

Easy 160i

Directions for use
Brugervejledning
Bruksanvisning
Brukerveiledning

The logo for QUICKIE, featuring a stylized, curved graphic above the word "QUICKIE" in a bold, sans-serif font. A registered trademark symbol (®) is located at the end of the word.

000 690 608.EMS

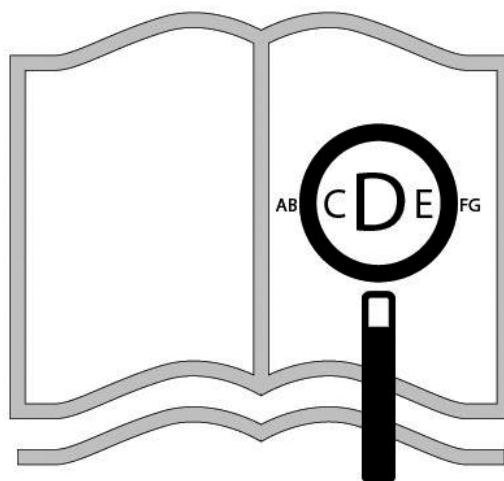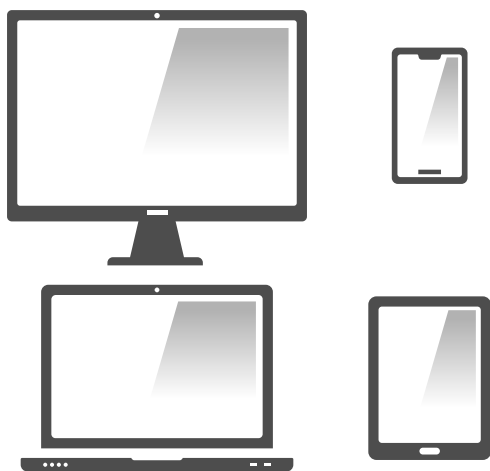www.sunrisemedical.com

If you are visually impaired, this document can be viewed in pdf format at **www.Sunrisemedical.co.uk** or alternatively is available on request in large text.

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Wheelchair Components

EN

We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R & D to production. This product complies with the standards set forth in EU and UK regulations. Options or accessories shown are available at extra cost.

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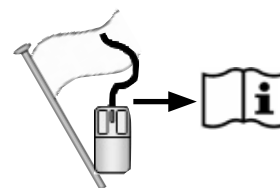
Foreword

Dear Customer,

We are very happy that you have decided in favour of a high-quality product from SUNRISE MEDICAL.

This owner's manual will provide numerous tips and ideas so that your new wheelchair can become a trustworthy and reliable partner in your life.

Maintaining close ties to our customers means a lot to us at Sunrise Medical. We like to keep you up-to-date about new and current developments at our company. Keeping close to our customers means: fast service, as little red tape as possible, working closely with customers. When you need replacement parts or accessories, or if you just have a question about your wheelchair – we are there for you.

We want you to be satisfied with our products and service. At Sunrise Medical we are constantly working to develop our products further. For this reason, changes can occur in our palette of products with regard to form, technology and equipment. Consequently, no claims can be construed from the data or pictures contained in this user's manual.

The management system of SUNRISE MEDICAL is certified to EN ISO 13485 and ISO 14001.

**UK
CA** As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the UK Medical Devices Regulation 2002 No. 618.

CE As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

Notice to the user and/or patient: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used.

Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.SunriseMedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, Annex I Nr. 14.1 of the Medical Device Regulation 2017/745.

Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk

Please contact your local, authorised SUNRISE MEDICAL dealer if you have any questions concerning the use, maintenance, or safety of your wheelchair.

In case there is no authorised dealer in your area, or you have any questions, contact Sunrise Medical either in writing or by telephone.

Sunrise Medical
Thorns Road
Brierley Hill
West Midlands
DY5 2LD
England
Phone: 0845 605 66 88
Fax: 0845 605 66 89
www.SunriseMedical.co.uk

IMPORTANT:

DO NOT USE YOUR WHEELCHAIR UNTIL THIS MANUAL HAS BEEN READ AND UNDERSTOOD. NOT FOLLOWING THESE INSTRUCTIONS MAY RESULT IN PHYSICAL INJURY, DAMAGE TO THE PRODUCT OR DAMAGE TO THE ENVIRONMENT!

Contents

1.0 General Safety Notes and Driving Limits	6
2.0 Transportation	8
3.0 Overview	10
4.0 Handling	11
5.0 Options	11
Quick-release axles	11
Step Tubes	11
Brakes	11
Footboard	12
Footrest	12
Headrests	13
Castor	13
Perforated plate	13
Rear seat height	13
Adjustable back sling	13
Height-adjustable backrest	14
Backrest angle	14
Rear Wheel Extension	14
Lap strap	14
Sideguard	14
Push Handle	14
Anti-tip tubes	15
Seat sling	15
Seat depth growth	15
Crutch holder	15
Tray	15
Stabilising bar	15
Transit wheels	15
Options – One Hand Operation	15
6.0 Tyres and Mounting	16
7.0 Trouble Shooting	16
8.0 Maintenance	16
9.0 Technical data	16
10.0 Nameplates / Guarantee	17
11.0 Torque	17

Use

Wheelchairs are exclusively for a user who is unable to walk or has limited mobility, for their own personal use in- and outdoor.

The maximum weight limit (includes both the user and any weight of accessories fitted to the wheelchair) is marked on the serial number label, which is affixed to the crossbar or stabiliser bar below the seat.

Warranty can only be taken on if the product is used under the specified conditions and for the intended purposes.

The intended lifetime of this product is 5 years.

Please DO NOT use or fit any 3rd party components to the wheelchair unless they are officially approved by Sunrise Medical.

Area of Application

Indications

The varieties of fitting variants, as well as the modular design, mean that it can be used by those who cannot walk or have limited mobility because of:

- Paralysis
- Loss of extremity (leg amputation)
- Extremity defect deformity
- Joint contractures/joint injuries
- Illnesses such as heart and circulation deficiencies, disturbance of equilibrium or cachexia, Neurological disease patterns, Muscle dystrophy, Hemiplegia as well as for elderly people who still have strength in the upper body.

Contraindications

The wheelchair shall not be used without attendant in case of:

- Perception disorder
- Imbalance
- Loss of both arms, if not supported by a caregiver
- Joint contracture or joint damage on both arms
- Seating disability

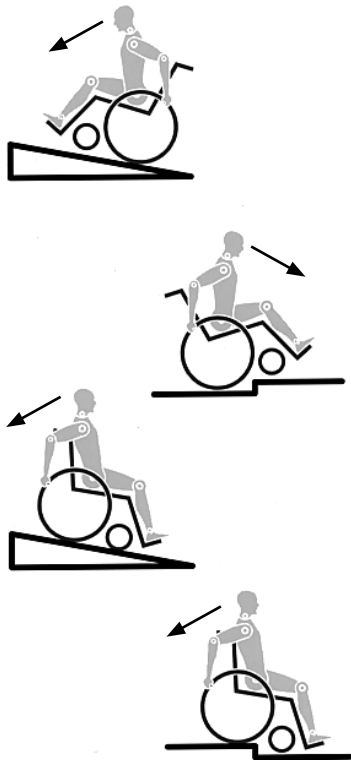
NOTE:

Please note that driving a wheelchair requires sufficient cognitive, physical and visual skills. The user must be able to assess the effects of actions during the operation of the wheelchair and, if necessary, to correct them. These capabilities and the safe use of the additionally attached components cannot be assessed by Sunrise Medical as a manufacturer. We cannot accept any liability for any damage resulting from this.

Please refer to the operating instructions of the wheelchair and the additionally mounted components. Instruct the user in the safe use of the wheelchair and the additionally mounted components. Inform users of specific warnings that need to be read, understood, and respected.

When considering provision, please also note the body size, weight, physical and psychological constitution, the age of the person, living conditions and environment.

1.0 General Safety Notes and Driving Limits

The engineering and construction of this wheelchair has been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded in parts. Nevertheless, users may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must be strictly observed. Unprofessional or erroneous changes or adjustments increase the risk of accident. As a wheelchair user, you are also part of the daily traffic on streets and pavements, just like anyone else. We would like to remind you that you are therefore also subject to any and all traffic laws.

Be careful during your first ride in this wheelchair. Get to know your wheelchair.

Before each use, the following should be checked:

- Quick-Release Axles for Rear Wheels
- Velcro on seats and backrests
- Tyres, tyre pressure, and wheel locks.

Before changing any of the adjustments of this wheelchair, it is important to read the corresponding section of the Owner's manual.

It is possible that potholes or uneven ground could cause this wheelchair to tip over, especially when riding uphill or downhill. When riding over a step or up an incline frontally, the body should be leaning forward.

NEVER exceed the maximum load of 140 kg for driver plus any items carried on the wheelchair. If you exceed the maximum load, this can lead to damage to the chair, or you may fall or tip over, lose control and may lead to serious injury of the user and other people.

When it is dark, please wear light clothing or clothing with reflectors, so that you can be seen more easily. Make sure that the reflectors on the side and back of the wheelchair are clearly visible. We would also recommend that you fit an active light.

To avoid falls and dangerous situations, you should first practice using your new wheelchair on level ground with good visibility.

When getting on or off the wheelchair, do not use the footboards. These should be flipped up beforehand and swung to the outside as far as possible. Always position yourself as close as possible to the place where you wish to sit.

Only use your wheelchair properly. For example, avoid travelling against an obstacle without braking (step, kerb edge) or dropping down gaps.

The wheel locks are not intended to brake your wheelchair. They are only there to ensure that your wheelchair does not begin rolling unintentionally. When you stop on uneven ground, you should always use the wheel locks to prevent such rolling. Always apply both wheel locks; otherwise, your wheelchair could tip over.

Explore the effects of changing the centre of gravity on the behaviour of the wheelchair, for example on inclines, slopes, all gradients or when overcoming obstacles. Do this with the secure aid of a helper.

With extreme settings (e.g. rear wheels in the most forward position) and less than perfect posture, the wheelchair may tip over even on a level surface.

Lean your upper body further forward when going up slopes and steps.

Lean your upper body further back when going down slopes and steps. Never try to climb and descend a slope diagonally.

Avoid using an escalator which may lead to serious injury in the event of a fall.

Do not use the wheelchair on slopes $> 10^\circ$. The Dynamic safe slope is dependant on the chair configuration, the users abilities and the style of riding. As the users abilities and style of riding cannot be pre-determined then the max safe slope cannot be determined. Therefore this must be determined by the user with the assistance of an attendant to prevent tipping. It is strongly recommended that inexperienced users have Ant-tips fitted.

It is possible that potholes or uneven ground could cause this wheelchair to tip over, especially when riding uphill or downhill.

Do not use your wheelchair on muddy or icy ground. Do not use your wheelchair where pedestrians are not allowed.

To avoid hand injuries do not grab in between the spokes or between the rear wheel and wheel lock when driving the wheelchair.

In particular when using lightweight metal handrims, fingers will easily become hot when braking from a high speed or on long inclines.

Only attempt stairs with the help of an attendant. There is equipment available to help you, e.g. climbing ramps or lifts, please use them. If there is no such equipment available, then the wheelchair must be tipped and pushed, never carried, over the steps (2 helpers). We recommend that users over 100 kg in weight do not use this stairway manoeuvre!

In general, any anti-tip tubes fitted must be set beforehand, so that they cannot touch the steps, as otherwise this could lead to a serious fall. Afterwards the anti-tip tubes must be set back to their correct position.

Make sure that the attendant only holds the wheelchair using securely mounted parts (e.g. not on the footrests or the sideguards).

When using the lifting ramp make sure that the anti-tip tubes fitted are positioned outside the danger area.

Secure your wheelchair on uneven ground or when transferring (e. g. into a car) by using the brakes.

If and whenever possible, during a journey in a specially fitted vehicle for disabled people, vehicle occupants should use the seats in the vehicle and the appropriate restraint system. This is the only way to ensure that occupants will have the maximum protection if there is an accident. When using safety elements offered by SUNRISE MEDICAL and using a specially designed safety system, lightweight wheelchairs can be used as a seat when being transported in a specially fitted vehicle. (See the Chapter on "Transportation").

Depending on the diameter and setting of the castors, as well as the centre of gravity setting of the wheelchair, the castors may begin to wobble at high speeds. This can lead to the castors being blocked and the wheelchair may tip over. Therefore, please make sure that the castors are adjusted correctly (see the Chapter "Castors"). In particular, do not travel on an incline without brakes, travel at a reduced speed. We recommend that novice users use anti-tip tubes.

Anti-tip tubes should prevent the chair tipping over backwards unintentionally. Under no circumstances should they take the place of transit wheels, and be used to transport a person in a wheelchair with the rear wheels removed.

When reaching for objects (which are in front of, to the side or behind the wheelchair) make sure that you do not lean too far out of the wheelchair, as if you change the centre of gravity there is a risk of tipping or rolling over. The hanging of additional load (back pack or similar items) onto your chair backposts can affect the rearward stability of your chair, especially when used in combination with recliner backrests. This can cause the chair to tip backwards causing injury.

For thigh amputees you must use anti-tip tubes.

Before setting off, check that your tyre pressure is correct. For rear wheels it should be at least 3.5 bar (350 kPa). The max. pressure is indicated on the tyre. The knee-lever brakes will only work if there is sufficient tyre pressure and if the correct setting has been made (see the Chapter on "Brakes").

If the seat and back sling are damaged, you must replace them immediately.

Be careful with fire, in particular with burning cigarettes. Seat and back slings can be set alight.

If the wheelchair is subject to direct sunlight for a long period of time, then parts of the wheelchair (e.g. frame, legrests, brakes and sideguard) may become hot (>41°C).

Always make sure that the quick-release axles on the rear wheels are set properly and lock in. If the button on the quick-release axle is not pressed in, the rear wheel cannot be removed.

WARNING!

The effect of the knee-lever brake as well as the general driving characteristics are dependant on tyre pressure. The wheelchair is significantly lighter and easier to manoeuvre when the rear wheels are pumped up correctly and both wheels have the same pressure.

WARNING!

Make sure that your tyres have sufficient tread! Please note that you are subject to any and all traffic laws when driving in public traffic.

WARNING!

Always be careful with your fingers when working or adjusting the wheelchair!

The wheelchairs shown and described in this user guide may not correspond in every detail exactly to your own model. However, all instructions are completely relevant, regardless of possible detail differences.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

We at SUNRISE MEDICAL have been awarded the ISO 13485 Certificate, which affirms the quality of our products at every stage, from R & D to production. This product complies with the standards set forth in EU directives. Optional equipment and accessories are available at extra charge.

2.0 Transportation

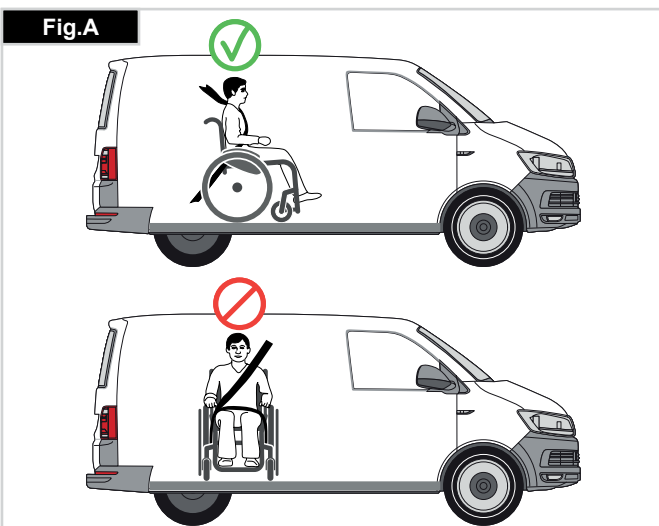
There is a risk of serious injury or death if this advice is ignored!

Transportation of your wheelchair within a vehicle:

A wheelchair secured in a vehicle will not provide the equivalent level of safety and security of a vehicle seating system. It is always recommended that the user transfers to the vehicle seating. It is recognised that this is not always practical for the user to be transferred and in these circumstances where the user must be transported whilst in the wheelchair the following advice should be followed:

Confirm that your chair is crashtest suitable (see nameplate or crashtest bracket at the rear of the chair (Fig. 1))

1. Confirm that the vehicle is suitably equipped to transport a passenger in a wheelchair, and ensure the method of access/egress is suitable for your wheelchair type. The vehicle should have the floor strength to take the combined weight of the user, the wheel chair and accessories.
2. Sufficient space should be available around the wheelchair to enable clear access to attach, tighten and release the wheelchair and occupant tie down restraints and safety belts.
3. The occupied wheelchair must be located in a forward facing position and secured by the wheelchair tie down and occupant restraint straps (WTORS tie downs meeting the requirements of ISO 10542 Part 2 or SAE J2249) in accordance with the WTORS manufacturer's instructions.
4. The wheelchair's use in other positions within a vehicle has not been tested e.g. transportation in a side facing position must not be carried out under any circumstances (Fig. A).
5. The wheelchair should be secured by a Tie Down Restraint system, conforming to ISO 10542 or SAE J2249 with non-adjustable front straps and adjustable rear straps, which typically use Karabiner clips/S hooks and tongue and buckle fittings. These restraints generally comprise of 4 individual straps that are attached to each corner of the wheelchair.
6. The tie-down restraint system should be fitted as shown on the following pages.
7. The tie-down restraints should be attached as close as possible at an angle of 45 degrees and tightened securely in accordance with the manufacturer's instructions.
8. Alterations or substitutions must not be made to the wheelchair tie down points or to structural and frame or components without consulting the manufacturer. Failure to do so will invalidate the ability of a Sunrise Medical wheelchair to be transported within a vehicle.

9. Both pelvic and upper torso restraint belts must be used to restrain the occupant to reduce the possibility of head and chest impacts with the vehicle components and serious risk of injury to the user and other vehicle occupants. (Fig. B) The upper torso restraint belt should be mounted to the vehicle "B" pillar - failure to do so will increase the risk of serious abdominal injuries to the user.

10. A head restraint suitable for transportation (see label on headrest) must be fitted and suitably positioned at all times during transportation.

11. Postural supports (lap straps, lap belts) should not be used or relied on for occupant restraint in a moving vehicle unless they are labelled as meeting the requirements specified in ISO 7176-19 or SAE J2249.

12. The safety of the user during transportation depends upon the diligence of the person securing the tie-down restraints and they should have received appropriate instructions and/or training in their use.

13. Wherever possible remove and stow safely away from the wheelchair, all auxiliary equipment, for example: Crutches, Loose cushions and Tray Tables.

14. Articulating/elevating leg rest should not be used in the elevated position when the wheelchair and user are being transported and the wheelchair is restrained using Wheelchair Transport and Occupant Restraints.

15. Reclining backrests should be returned to an upright position.

16. The manual brakes must be firmly applied.

17. Restraints should be mounted to the vehicle "B" pillar and should not be held away from the body by wheelchair components such as armrest or wheels.

13. Wherever possible remove and stow safely away from the wheelchair, all auxiliary equipment, for example: Crutches, Loose cushions and Tray Tables.

14. Articulating/elevating leg rest should not be used in the elevated position when the wheelchair and user are being transported and the wheelchair is restrained using Wheelchair Transport and Occupant Restraints.

15. Reclining backrests should be returned to an upright position.

16. The manual brakes must be firmly applied.

17. Restraints should be mounted to the vehicle "B" pillar and should not be held away from the body by wheelchair components such as armrest or wheels.

Occupant Restraints Instruction

1. The pelvic restraint belt must be worn low across the front of the pelvis so that the angle of the pelvic belt is within the preferred zone of 30 to 75 degrees to the horizontal.

A steeper (greater) angle within the preferred zone is desirable i.e. closer to, but never exceeding 75 degrees. (Fig. C)

2. The upper torso restraint belt must fit over the shoulder and across the chest as illustrated Fig d and e. Restraint belts must be adjusted as tightly as possible consistent with user comfort.

Restraint belt webbing must not be twisted when in use.

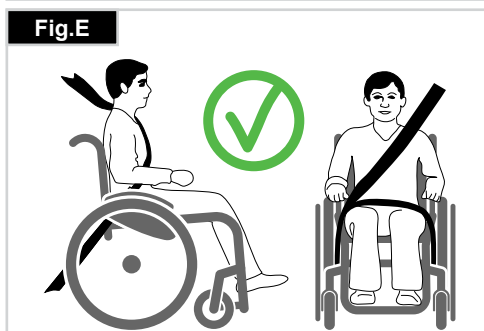
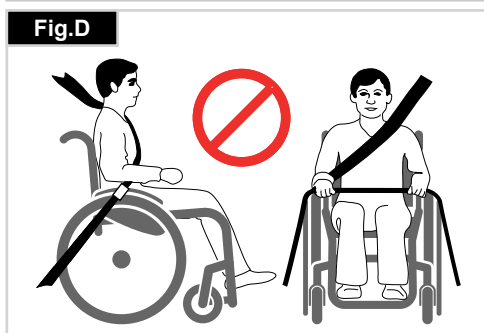
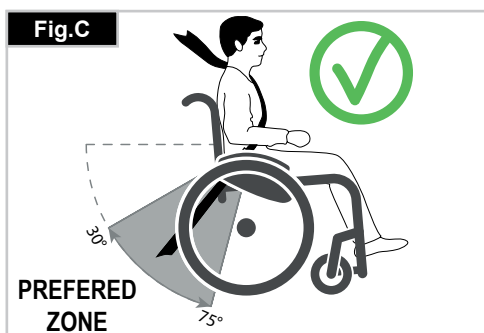
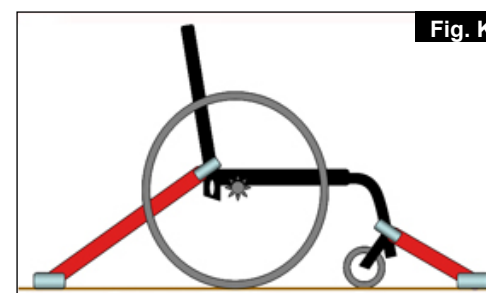
The upper torso restraint belt must fit over the shoulder and across the shoulder as illustrated in Fig D and E.

3. The attachment points to the chair are the inner front side frame just above the castor, and the rear side frame. The straps are fitted around the side frames at the intersection of the horizontal and vertical frame tubes. (See Fig. G-H-I)

4. The tie down symbol (Fig F) on the wheelchair frame indicates the position of the wheelchair restraint straps. The straps are then tensioned after the front straps have been fitted to secure the wheelchair.

Positioning of wheelchair tie down restraints on wheelchair:

1. Location of the front and rear tie down labels (Fig. G - H).
2. Position of the front (Fig. I) and rear (Fig. J) wheelchair tie down restraint and the tie down label.
3. Side view of tie down straps (Fig. K).

3.0 Overview

1. Push handle
2. Sling backrest
3. Sideguard
4. Seat sling
5. Footboard
6. Castor
7. Castor adapter
8. Castor fork
9. Quick-release axle
10. Brake
11. Handrim
12. Drive wheel

4.0 Handling

Folding up

First remove the seat cushion from the wheelchair and flip-up the footboard (continuous) or footboards.

Take hold of the sling or the seat tubes in the middle, from below and pull upwards. The chair will fold up.

To fold your wheelchair so that it is as small as possible, e.g., to put into a car, you can remove the footrests (depending on the model). To do this, push the release lever from the outside and swing the footrest to the side, then pull the footrest out of the frame tube.

Fig. 4.1

Unfolding

Before unfolding, undo the folding bracket

Press down on the seat tube (see figure). The wheelchair will unfold. Then press the seat tube into the support. This process is easier if you tip the chair a little, so that there is no load on one of the rear wheels and then the chair will unfold more easily.

Fig. 4.2

ATTENTION!

Risk of trapping fingers

5.0 Options

Quick-release axles

Quick-release axles on rear wheel

The rear wheels are equipped with quick-release axles. The wheels can, therefore, be installed or removed without using tools. To remove a wheel, simply depress the quick-release button on the axle (1) and pull it out.

Fig. 4.3

ATTENTION!

Hold the quick-release button on the axle depressed when inserting the axle into the frame to mount the rear wheels. Release the button to lock the wheel in place. The quick-release button should snap back to its original position.

Step Tubes

Step Tubes

Step tubes are used by attendants to push a wheelchair over an obstacle. Simply step on the tube to push a wheelchair, for example, over a kerb or step.

Fig. 4.4

Brakes

Wheel locks

Your wheelchair is equipped with two wheel locks. They are applied directly against the tyres.

To engage, press both wheel lock levers forward against the stops (Fig. 4.5 and 4.7). To release the wheel lock, pull the levers back to their original positions.

Braking power will decrease with:

- Worn tyre tread
- Tyre pressure that is too low
- Wet tyres
- Improperly adjusted wheel locks

The wheel locks have not been designed to be used as brakes for a moving wheelchair. The wheel locks should therefore never be used to brake a moving wheelchair. Always use the handrims for braking. Make sure that the interval between the tyres and wheel locks complies with the given specifications (Fig. 4.6 and 4.8). To re-adjust, loosen screw (1) and set the appropriate interval. Tighten screw (see the page on torque).

ATTENTION!

After each adjustment of the rear wheels, check the interval to the wheel locks and readjust if necessary.

Fig. 4.5

4.5 Knee lever brake

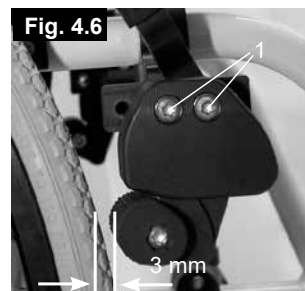
Fig. 4.6

4.6 Knee lever brake

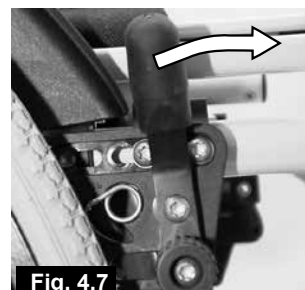
Fig. 4.7

4.7 Standard brake

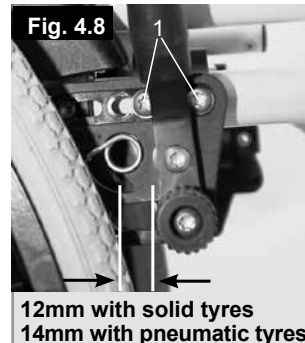
Fig. 4.8

12mm with solid tyres
14mm with pneumatic tyres

4.8 Standard brake

Brake lever extension

The longer lever helps to minimise the effort needed to set the wheel locks. The brake lever extension is screwed to the brakes. By raising this, it can be flipped forward.

ATTENTION!

Incorrect mounting of the wheel lock will result in a higher effort to operate. This might cause the wheel lock extension lever to break.

Leaning onto the wheel lock extension lever while transferring will cause the lever to break. There is a risk it could break. Splashing water from tyres might cause the wheel lock to malfunction.

Fig. 4.9

Drum brake

Drum brakes permit safe and convenient braking for an attendant. They can also be applied with the aid of a locking lever (1). The locking lever must snap audibly into place. Performance of the drum brakes is not dependant on the air pressure inside the tyres. The wheelchair cannot be pushed when the drum brakes are set.

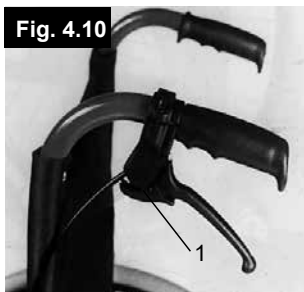
Fig. 4.10

⚠ ATTENTION!

Drum brakes should only be adjusted by authorised dealers.

One-hand operation

The wheelchair is equipped with two wheel locks, which can either be used both together or from each side individually. They are applied directly against the tyres. To engage, press the wheel lock lever forward against the stop (Fig. 4.12). To release the wheel lock, pull the levers back to their original positions. The one-arm brake is always delivered with a brake lever extension. The longer lever helps to minimise the effort needed to engage the wheel locks. The brake lever extension is screwed to the brakes. By raising this, it can be flipped forward.

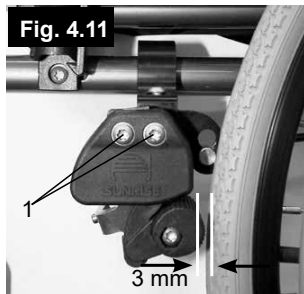
Fig. 4.11

Fig. 4.12

Braking power will decrease with:

- Worn tyre tread
- Tyre pressure that is too low
- Wet tyres
- Improperly adjusted wheel locks

The wheel locks have not been designed to be used as brakes for a moving wheelchair. The wheel locks should therefore never be used to brake a moving wheelchair. Always use the handrims for braking.

Make sure that the interval between the tyres and wheel locks complies with the given specifications (Fig. 4.11). To re-adjust, loosen screw (1) and set the appropriate interval. Tighten screw (see the page on torque).

⚠ ATTENTION!

After each adjustment of the rear wheels, check the interval to the wheel locks and readjust if necessary.

Incorrect mounting of the wheel lock will result in a higher effort to operate. This might cause the wheel lock extension lever to break. Leaning onto the wheel lock extension lever while transferring will cause the lever to break. There is a risk it could break. Splashing water from tyres might cause the wheel lock to malfunction.

Footboard

Footboard

There are several footboards available. These can be flipped up to make it easier to transfer to/from the chair.

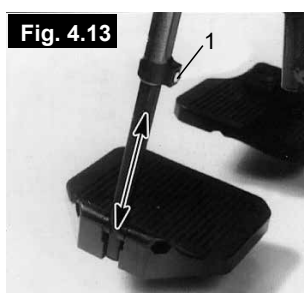
Fig. 4.13

Lower Leg Length

By undoing the clamp screw (1), you can adjust the lower leg length. Loosen the clamp screw, adjust the tube with the footboard to the desired position. Tighten clamp screws again (see the page on torque). Ensure that the plastic push joint is located under the clamp screw in the correct position. Ensure that the plastic push joint is located under the clamp screw in the correct position.

Angle-adjustable footboard

The footboard can be flipped up to make it easier to transfer to/from the chair.

It can be adjusted to alter the angle to the ground. Tighten screw (1) on the outside. By loosening the clips (2) the footboard can be moved to three positions both forwards and backwards. By adjusting the set screw (3) the footboard can be changed to a horizontal position. To do this, the footboard must be flipped up. Make sure that after any adjustment work, all screws are tightened correctly (see the page on torque). Ensure that you maintain the minimum gap to the floor (25 mm).

Fig. 4.14

Fig. 4.15

Fig. 4.16

Footrest

Footrest and swing away latch

When fitting the footrests these are fitted in the swung-away position. Then simply turn them inwards until the interlock engages (1). To remove the footrests, use the lever (2) and swing the footrest outwards and lift off. Check that the footrest is correctly engaged.

⚠ ATTENTION!

The footrests are not designed to be used to lift or carry the wheelchairs with an occupant.

The footrest can be adjusted easily using eccentric nuts. You can change the setting as follows: Release the screw. Rotate the eccentric nut. Check that the lock clicks into place and releases correctly. To fit the eccentric bolts firmly, hold with the wrench and tighten the screw (see the page on torque).

Fig. 4.17

Flip-up footrest - standard or with length compensation

To flip up:

Pull the footrest upwards and adjust to the desired height. When doing this, the footrest clicks audibly into place.

To lower:

Briefly take the load off the footrest by lifting the lower leg and then push the release lever. In this way you can swing the footrest downwards. As soon as you let go of the lever, the footrest will click back into place.

Risk of trapping fingers!

When moving the footrest up or down, do not put your fingers in the adjusting mechanism between the frame and the moving parts of the footrest.

The footrests are not designed to be used to lift or carry the wheelchairs with an occupant.

Headrests

Headrests

The headrest can be raised and moved horizontally both forwards and backwards. To do this, loosen the screw (1 or 2). Now you can adjust to the desired position. Tighten screws again (see the page on torque).

Castor

Castor, Castor adapter, Castor fork

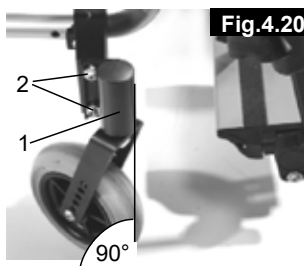
From time to time the wheelchair may veer slightly to the right or left, or the castors may flutter. This may be caused by the following:

- The forward or reverse wheel motion has not been set properly.
- The castor angle has not been adjusted properly.
- Castor and/or rear wheel air pressure is incorrect; wheels do not turn smoothly.

The optimum adjustment of the castors is required so that the wheelchair runs in a straight line. The castors should always be adjusted by an authorised dealer.

The castor plates must be re-adjusted, and the wheel locks must be checked any time the rear wheel position has been altered.

Check the setting of the castor adapter (1) by placing a set-square next to it and looking at it from the front. Make sure that the adapter is at right angles to the floor. By undoing the screws (2) and moving the adapter, you can then re-adjust the wheels to their optimum position, by adjusting the corresponding hex nuts. Make sure that after any adjustment work, all screws are tightened correctly (see the page on torque).

Front seat height Easy 160

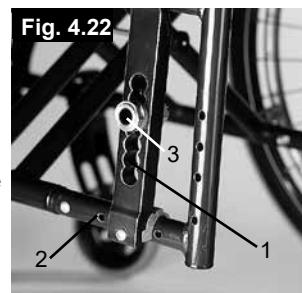
To adjust the front seat height, please undo the screws A (2 on each side) and adjust to the desired seat height. Make sure that both sides are at the same height, and then re-tighten the screws. Please observe the Nm torque value (see the page on torque).

Perforated plate

Perforated plate (only Germany)

The position of the centre of gravity in relation to the rear wheel is an important factor for easy and comfortable driving. Several positions are possible. If you move the perforated plate to the rear (1) in the frame holes (2), the chair is in a safe position, but at the same time is not so manoeuvrable, as it is when the perforated plate is positioned further forward. By moving the axle adapter (3) in the perforated plate, the seat height of the chair can be adjusted. Make sure that after any adjustment work, all screws are tightened correctly (see the page on torque).

If necessary, the brakes and castors must be re-adjusted.

Rear seat height

Rear seat height

Loosen the 4 screws (2 on each side) and remove them from the axle plate. Then move the axle adapter to the desired height and after that, re-tighten the screws. Make sure that both sides are set to the same position. Please observe the Nm torque value (see the page on torque).

Adjustable back sling

Adjustable back sling

The adjustable back sling can be adjusted for tension by using several straps. The back sling upholstery can be accessed from the inside via an opening and can be padded to suit individual tastes.

Height-adjustable backrest

Height-adjustable backrest Easy 160

Loosen screw B and push the backrest tube C to the desired height. Then re-tighten the screw B. Please observe the Nm torque value (see the page on torque). Adjust both sides in the same way and then, using the Velcro fasteners, set the sling to the desired position.

Backrest angle

Backrest angle

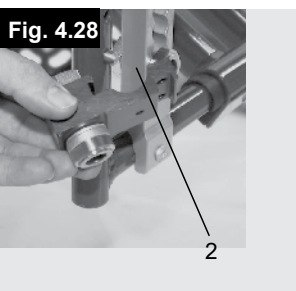
Undo the screw A and pull the screw out of its position. Set the desired angle and then re-tighten the screw. Make sure that the angle is set the same on both sides. Please observe the Nm torque value (see the page on torque).

Rear Wheel Extension

Rear Wheel Extension

The larger the wheelbase the greater security against your wheelchair tipping over. To do this, screw the axle adapter (1) to the frame at the desired height. Please ensure that the steel pin (2) is correctly fitted into the holes provided in the frame. Make sure that after any adjustment work, all screws are tightened correctly (see the page on torque).


ATTENTION!

Wheel locks must be adjusted to match the new position!

Lap strap

Lap strap

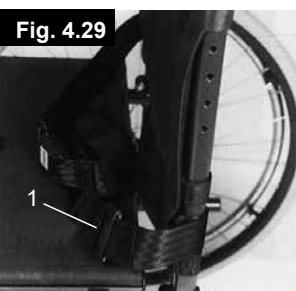The lap belt enhances wheelchair safety. The lap belt is fastened to the frame as shown in the picture.

Sideguard

Sideguards - clothes protector

The clothes protector prevents clothes getting dirty from spray water. You can set the position in relation to the rear wheel by moving the sideguard. To do this, remove the screws (1 and 2). After setting to the desired position, re-tighten the screws (see the page on torque).

ATTENTION!

Neither the side-guards nor the armrests are to be used for lifting or carrying the wheelchair.

Standard sideguard, flip-up, removeable with short or long armrests

The sideguard with shaped front edges will allow you to get up close to a table. To flip the armrest up, push the lever (1) forwards, so that the sideguard is released. The height of the armrest (2) can be adjusted by fitting the sockets (3) at different positions.

To do this release the screws (4), move the sockets (3) and then re-tighten the screws (4) (see the page on torque).

ATTENTION!

Neither the side-guards nor the armrests are to be used for lifting or carrying the wheelchair.

Sideguard, flip-up, removeable with short or long armrests, height-adjustable

The armrest can be height-adjusted as follows.

Push the lever (1) upwards and move the armrest (2) to the desired height. Release the lever and press the armrest (2) downwards until you hear it click into place. Always check that the sideguards are correctly engaged. To flip the armrest up, pull the lever (3) so that the sideguard is released.

ATTENTION!

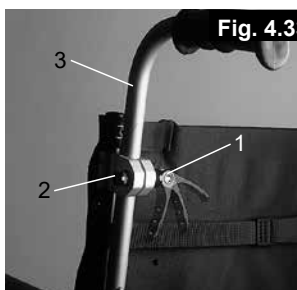
Neither the sideguards nor the armrests are to be used for lifting or carrying the wheelchair.

RISK OF TRAPPING FINGERS: When using a 24" (609.6mm) rear wheel the armpad must be moved one level up.

Push Handle

Height-adjustable push handles

These handles are secured with pins to prevent them from sliding out unintentionally. Opening the quick-release lever (1) makes it possible to adjust the height of the push handles (3) to meet your individual needs. As you move the lever, you will hear a locking mechanism; you may now easily position the push handle as desired. The nut (2) on the tension lever determines how tightly the push handles are clamped into place. If the nut is loose after adjusting the tension lever, the push handle will also be too loose. Turn the push handle from side to side before use to make sure that it is clamped securely enough into place. After adjusting the handle height, always clamp the tension lever (1) securely into place. If the lever is not secure, injuries could result when ascending stairs.

Anti-tip tubes

Anti-tip tubes

The anti-tip tubes (1) provide additional safety for inexperienced users when they are still learning how to operate their wheelchairs. Anti-tip tubes (1) prevent a wheelchair from tipping over backwards. The anti-tip tube (1) can be swung downwards by pushing it or rotated forwards. There should be a gap of 30 mm to 50 mm to the floor. To go up or down a step (e.g. kerb) the anti-tip tubes must be swung forwards, to prevent them from touching the ground.

Seat sling

Standard seat sling

The seat sling has a Velcro fastening on one side. This allows you to infinitely adjust the seat sling. To re-adjust the tension of the seat sling, fold the wheelchair slightly. Remove the screws (1) and pull the front plugs (2) forwards out of the frame. Then the seat sling (3) can be pulled forward out of the frame. By unfastening the Velcro, the seat sling can now be re-adjusted. Follow the instructions in the opposite direction to refit the seat sling. Make sure that after any adjustment work, all screws are tightened correctly (see the page on torque).

ATTENTION!

In order to maintain safety, at least 50% of the Velcro surfaces must be in contact with each other.

Seat depth growth

Seat depth growth

You can easily alter the seat depth by using the Velcro fastener on the seat sling. Separate sling A from sling B, set the desired position and then push both slings back together, one on top of each other.

Crutch holder

Crutch holder

This device permits crutches to be transported directly on the wheelchair. It has a Velcro loop to fasten crutches or other aids.

ATTENTION!

Never try to use or even remove the crutches or other aids while moving.

Tray

Tray

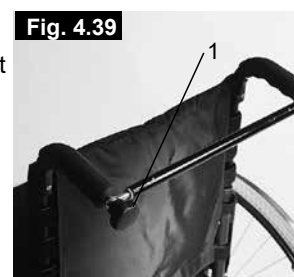
The tray provides a flat surface. The tray must be adjusted once to the width of the seat by an authorised dealer. When carrying out a functional test, the user must be sitting in the wheelchair.

Stabilising bar

Stabilising bar

The stabilising bar gives the backrest its stability. To be able to fold the chair, you must turn the knobs (1) on the end of the push handle and remove the stabilising bar, then you can fold the wheelchair easily. To replace the stabilising bar, you must retighten the knobs, so that it is locked in place.

Transit wheels

Transit wheels

Travel wheels should be used wherever your wheelchair would be too wide if the rear wheels were used. After the rear wheels have been removed with the aid of the quick-release axles, the transit wheels can immediately be used to continue riding. The transit wheels are mounted so that they are approx. 30 mm above the ground when not in use. They are thus out of the way when riding, transporting, or when tipping to pass over obstacles.

ATTENTION!


Without rear wheels, your wheelchair has no wheel locks.

Options – One Hand Operation


One Hand Operation

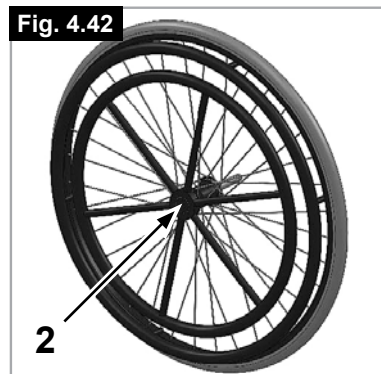(Fig. 4.42 - 4.42.1)
To propel the chair straight forward both handrims must be used. To fold the wheelchair disconnect the telescopic bar by pushing it inward (1). To take off the wheels, press the locking button on the axle (2).

CAUTION!

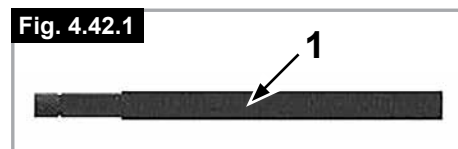Make always sure that the connections are set properly to avoid injuries.

6.0 Tyres and Mounting

Tyres and Mounting

Always make sure that you maintain the correct tyre pressure, as this can have an effect on wheelchair performance. If the tyre pressure is too low, rolling resistance will increase, requiring more effort to move the chair forward. Low tyre pressure also has a negative impact on manoeuvrability. If the tyre pressure is too high, the tyre could burst.

The correct tyre pressure for a given tyre is printed on the surface of the tyre.

Tyres can be mounted the same way as an ordinary bicycle tyre.

Before installing a new inner tube, you should always make sure that the base of the rim and interior of the tyre are free of foreign objects. Check the pressure after mounting or repairing a tyre. It is critical to your safety and to the wheelchair's performance that regulation air pressure be maintained and that tyres are in good condition.

7.0 Trouble Shooting

Wheelchair pulls to one side

- Check tyre pressure
- Check to make sure wheel turns easily (bearings, axle)
- Check angle of castors
- Check to make sure both castors are making proper contact with the ground

Castors begin to wobble

- Check angle of castors
- Check to make sure all bolts are secure; tighten if necessary (see the page on torque)
- Check to make sure both castors are making proper contact with the ground

Wheelchair does not fit in the support

- The chair is still new, or the seat or back slings are still stiff, this will ease through frequent use

Wheelchair is difficult to fold up

- Adjustable backrest sling is too stiff, loosen slightly.

Wheelchair squeaks and rattles

- Check to make sure all bolts are secure; tighten if necessary (see the page on torque)
- Apply small amount of lubrication to spots where movable parts come in contact with one another

Wheelchair wobbles

- Check angle of castors
- Check tyre pressure
- Check to see if rear wheels are adjusted differently

8.0 Maintenance

Maintenance

- Check the tyre pressure every 4 weeks. Check all of the tyres for wear or damage.
 - Check the brakes approximately every 4 weeks to make sure that they are working properly and easy to use.
 - Tyres can be mounted the same way as an ordinary bicycle tyre.
 - All of the joints that are critical to using your wheelchair safely are fitted with self-locking nuts. Check every 3 months to make sure that all bolts are secure (see the page on torque).
 - Self-locking nuts should only be used once and should be replaced after single use.
 - Use only mild household cleansers when your wheelchair is dirty. Use only soap and water when cleaning the seat upholstery.
 - If your wheelchair should ever get wet, please dry it after use.
 - A small amount of sewing-machine oil should be applied to quick-release axles approximately every 8 weeks.
- Depending on the frequency and type of use, we recommend taking your wheelchair to your authorised dealer every 6 months, to have it inspected by trained personnel.

Hygiene measures when being re-used:

Prior to the wheelchair being re-used, it must be carefully prepared. All surfaces which come into contact with the user must be treated with a disinfection spray.

To do this, you must use a disinfectant as authorised/recommended in your country, for rapid alcohol-based disinfection for medical products and medical devices, which must be disinfected quickly.

Please be aware of the manufacturer's instructions for the disinfectant you are using.

In general, a complete disinfection cannot be guaranteed on seams. We therefore recommend that you dispose of seat and back slings to avoid micro-bacterial contamination with active agents according to your local infection protection law.

ATTENTION!

Sand and sea water (or salt in the winter) can damage the bearings of the front and rear wheels. Clean the wheelchair thoroughly after exposure to these conditions.

9.0 Technical data

Overall width:

- With standard wheels including handrims, close mount: SW + 180 mm
- With drum brake wheels including handrims, close mount: SW + 230 mm

Folded dimensions:

- with rear wheels: approx. 300 mm
- without rear wheels: approx. 220 mm

Weight in kg:

from 13.1 kg

Maximum load:

Approved to load of 140 kg

10.0 Nameplates / Guarantee

Nameplate

The name plate is located on either the cross-tube or the frame cross tube and also on the back page of the user's manual. The name plate indicates the exact model designation as well as other technical specifications. Please provide the following information whenever you have to order replacement parts or to file a claim

- Serial Number
- Order number
- Month/Year

SAMPLE

		Sunrise Medical GmbH Kahlbachring 2-4 D-69254 Malsch / Germany		2021-04-09	
TYPE:		Rollstuhl		SN 203211733451939	
Easy 160i		ISO 7176-19:2008			
140 kg	max 10°	540 mm	540 mm		

UK	RP	Sunrise Medical Limited Thorns Road, Brierley Hill West Midlands, DY5 2LD UNITED KINGDOM
-----------	-----------	---








TYPE:	Product Name/SKU Number.
	Maximum safe slope with anti-tip tubes fitted, Depends on wheelchair setting, posture and physical capabilities of the user.
	Seat width.
	Seat depth.
	Load Maximum.
	UKCA Mark.
	CE Mark.
	Consult instructions for use.
XXXX-XX-XX	Date of manufacture.
	Serial number.
	This symbol means Medical Device.
	Manufacturer's address.
ISO 7176-19:2008	Crash tested according to ISO 7176-19:2008.
	Importer's address
UK RP	UK Responsible Person
CH REP	Swiss Representative's address

Guarantee

We are very happy that you have decided in favour of a high-quality product from Sopur. As a sign of our gratitude, we are providing you with a 5-year guarantee on all frame components and on the cross-tube assembly as well as 1 year on all other components. We are not responsible for any damage resulting from inappropriate or unprofessional installation and/or repairs, through neglect and wear, or from changes in any wheelchair components caused either by the user or by third parties. In such cases, this guarantee shall be considered null and void.

Personalised wheelchairs cannot be exchanged.

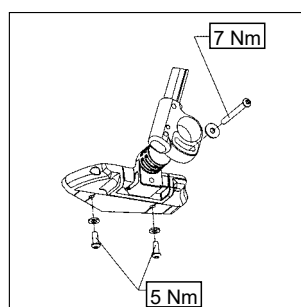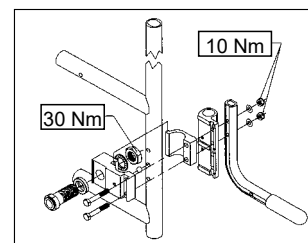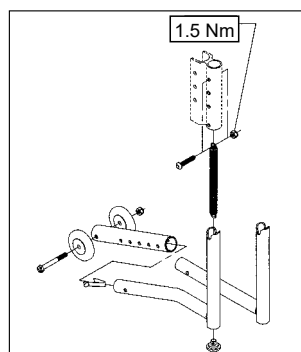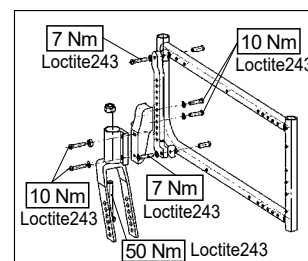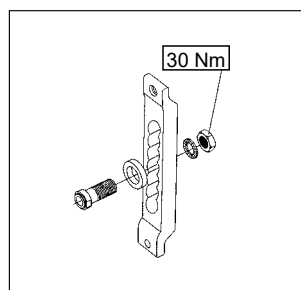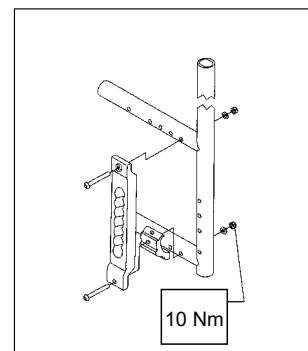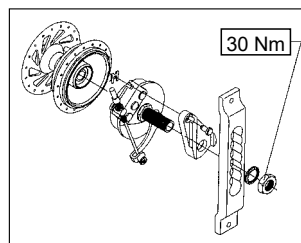11.0 Torque

Torque for M6 screws is 7 Nm, unless otherwise specified.

ISO 7010-M002
Instruction manual/booklet
must be read! (Blue Icon)

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Sunrise Medical HCM B.V.
Sunrise Medical SAS