



Technical / News Bulletin

UK

BULLETIN NO.:

TB 0440

DATE ISSUED

30/10/14

BULLETIN DETAILS

Product Group: Power

Model Affected: Quickie Salsa and Jive families

Effective Date: 10/06/2014

Serial No. Effective From: N/A

Bulletin Subject: Tilt creep switch protection on lift / tilt module

Subject of Bulletin:

As part of Sunrise Medicals commitment to improving our products and listening to quality and design feedback from our customer base, an additional protection cover has been added to the creep switch fitted to the new version of lift / tilt module.

Feedback from the field highlighted that the creep switch on the new version of lift / tilt module was exposed and subject to possible physical and environmental damage. To resolve this, a metal cover has been added to protect the switch mechanism. The metal cover is bolted on using the existing microswitch fixing screws, and hence can be retro-fitted if required.



Unprotected switch



New protection on switch – Part code 21012062

Literature Affected:

Sunparts: Updated with new part number – see parts.sunrisemedical.eu

Order Forms: No change

Website: No change

Owners Manual: No change

If you have any questions regarding this bulletin, please do not hesitate to contact:

Customer Services	Customer Support	Technical Service Centre	E-mail
Telephone:	+44 (0)845 605 66 88 Option 2	+44 (0)845 605 66 88 Option 3	enquiries@sunmed.co.uk
Fax:	+44 (0)845 605 66 89	+44 (0)845 605 66 89	help.technical@sunmed.co.uk