



# Technical / News Bulletin

UK

BULLETIN NO.:

**TB 0459**

DATE ISSUED

04-02-2015

## BULLETIN DETAILS

**Product Group:** Manual Wheelchairs

**Model Affected:** Quickie Life Folding

**Effective Date:** Immediate

**Serial No. Effective From:** Implemented since September 2014


**Bulletin Subject:** Improvement to cross brace

### Description of issue

A number of Life folding chairs in the field would not unfold correctly - the seat rails would not slide into the receivers.

### Solution – Refer to image below

The new cross brace has an increased diameter, moving from 23mm to 25mm for the single tube, and reduced tolerances for the central bolt to remove the folding issues

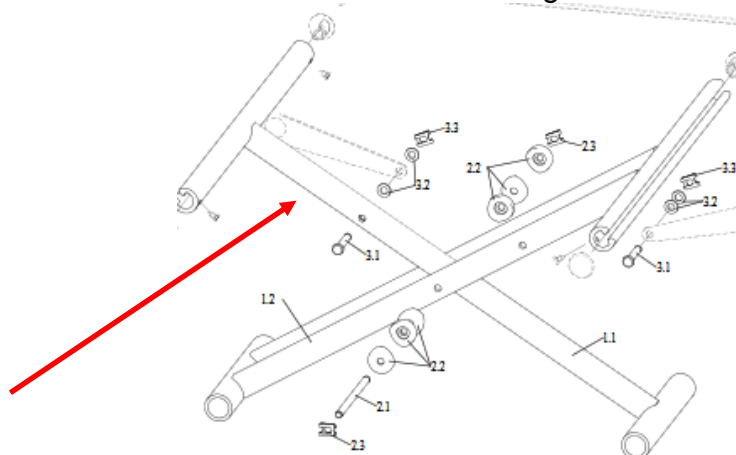

### Benefits to customer

1. Improved folding and unfolding performance
2. Increased rigidity.

### Literature Affected:

<b>Sunparts:</b>	Running change. No updates required
<b>Order Forms:</b>	No changes required
<b>Website:</b>	No changes required
<b>Owners Manual:</b>	No changes required

If you have any questions regarding this bulletin, please do not hesitate to contact:

Customer Services	Customer Support	Technical Service Centre	E-mail
<b>Telephone:</b>	<b>+44 (0)845 605 66 88</b> <b>Option 2</b>	<b>+44 (0)845 605 66 88</b> <b>Option 3</b>	enquiries@sunmed.co.uk
<b>Fax:</b>	<b>+44 (0)845 605 66 89</b>	<b>+44 (0)845 605 66 89</b>	help.technical@sunmed.co.uk