



BULLETIN DETAILS

Product Group:	Manual	
Model Affected:	Rubix ² , Helix ² and Helix ² Comfort	
Effective Date: 2nd March 2016	Serial No. Effective From:	N/A
Bulletin Subject:	Product improvement for Rubix ² , Helix ² and Helix ² Comfort	

Description of Bulletin:

As part of continuous efforts to look for new innovative ways to develop our product range and increase the possible adjustments, we have now added a fifth hole in the castor fork of the Breezy Rubix², Helix², Helix² and Comfort.

Benefits to the Customer

By adding the fifth hole to the castor forks there is now a greater range of options for the front seat height (see image). Therefore this means you can cater for a wider range of clients. Also due to this fifth hole, the increments between each adjustment have decreased which means the usability of these products are great and also makes the client more comfortable.

To see the range of the new options please compare the following tables.

Front seat height in cm

	37.5	39	40	41	42.5	44	45	46.5	48	49	50.5	52*	53*
6" - 12"							○	○	○				
7" - 12"							○	○	○	○			
8" - 12"							○	○	○	○	○	□	□
6" - 22"	○	○	○	○	○	○	○	○	○				
7" - 22"				○	○	○	○	○	○	○			
8" - 22"					○	○	○	○	○	○	○	□	□
6" - 24"	○	○	○	○	○	○	○	○	○				
7" - 24"				○	○	○	○	○	○	○			
8" - 24"					○	○	○	○	○	○	○	□	□

New Seat Height Options

Front seat height in cm

Castor & Wheel size	37.5	40	41.5	42.5	44	45	46.5	48	49	50.5	53**
6" Castor / 12" rear wheel						○					
7" Castor / 12" rear wheel					○		○				
8" Castor / 12" rear wheel						○					
6" Castor / 22" rear wheel	○	○		○		○		○			
7" Castor / 22" rear wheel			○		○		○		○		
8" Castor / 22" rear wheel				○		○				○	
6" Castor / 24" rear wheel		○		○		○		○			
7" Castor / 24" rear wheel			○		○		○		○		
8" Castor / 24" rear wheel				○		○		○		○	○

Previous Seat Height Options

Literature affected:

- Owner's Manual To be updated

- Order Form To be updated

If you have any questions regarding this bulletin, please do not hesitate to contact:

Customer Services	Customer Support	Technical Service Centre	E-mail
Telephone:	+44 (0)845 605 66 88 Option 2	+44 (0)845 605 66 88 Option 3	enquiries@sunmed.co.uk
Fax:	+44 (0)845 605 66 89	+44 (0)845 605 66 89	help.technical@sunmed.co.uk