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Nuage PLS

NUAGE PLS
NUAGE PLS ACTIVE
NUAGE PLS ACTIVE XS

USER MANUAL
GEBRUIKERSHANDLEIDING
NOTICE D'UTILISATION

EN
NL
FR

 **Breezy®**

9011307E

English.....3

Nederlands31

Français.....59

English

© 2019 Sunrise Medical

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of Sunrise Medical.

The information provided is based on general data concerning the construction known at the time of the publication of this manual. Sunrise Medical executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Sunrise Medical can therefore not be held liable for any damage resulting from specifications of the product deviating from the standard configuration. Illustrations contained in the manual may deviate from the configuration of your product.

The information made available has been prepared with all possible diligence, but Sunrise Medical cannot be held liable for any errors contained in the information or the consequences thereof.

Sunrise Medical accepts no liability for loss resulting from work executed by third parties.

Names, trade names, trademarks etc. used by Sunrise Medical may not, as per the legislation concerning the protection of trade names, be considered as being available.

Table of contents

1	Introduction.....	6
1.1	This user manual	6
1.2	For further information	6
1.3	Symbols used in this manual	6
2	Safety	7
2.1	Temperature	7
2.2	Moving parts	7
2.3	General safety guidelines and instructions	7
3	Product description.....	8
3.1	Nuage PLS series	8
3.2	Positioning options and accessories	9
4	Use	10
4.1	Getting into and out of the seat	10
4.2	Reducing the seating system in size for transportation	10
4.3	Use as a passenger seat in a vehicle	10
4.4	Adjusting the seating system	10
5	Settings (by a professional).....	11
5.1	Webbing straps and contour cushion (II)	11
5.2	Seat and backrest shells (II)	12
5.3	Seat webbing (II)	12
5.4	Contour cushion (II)	12
5.5	Pelvic obliquity (II)	13
5.6	Leg length discrepancy (II)	14
5.7	Backrest webbing (II)	14
5.8	Dorsal obliquity (II)	14
5.9	Tensioning the cover	15
5.10	Cushion filling	15
5.11	Seat width (wheelchair specific)	16
5.12	Seat depth (wheelchair specific)	16
5.13	Backrest width	17
5.14	Backrest height (wheelchair specific)	17
5.15	Funnel shape backrest shells (wheelchair specific)	17
5.16	Angle backrest shells (wheelchair specific)	18
5.17	Backrest angle (wheelchair specific)	18
5.18	Webbing straps (I)	18
5.19	Seat and backrest shells (I)	19
5.20	Seat webbing (I)	19
5.21	Pelvic obliquity (I)	19
5.22	Leg length discrepancy (I)	20
5.23	Contour cushion (I)	20
5.24	Backrest webbing (I)	20
5.25	Dorsal obliquity (I)	21

6	Maintenance.....	22
6.1	Cleaning the upholstery	22
6.2	Replacing the upholstery	22
7	Troubleshooting.....	23
8	Technical specifications.....	24
8.1	CE declaration and requirements	24
8.2	Technical information	24
9	Warranty.....	26
9.1	Definitions of terms	26
9.2	Warranty period table	27
10	Labelling.....	28
10.1	Markings on the seating system	28
11	Accessories.....	29

1 Introduction

Congratulations on choosing a Nuage PLS seating system for your tilt in space wheelchair. All of our high-quality mobility products are designed to enhance independence and make your everyday life easier.

1.1 This user manual

This user manual will help you to use and maintain your seating system safely. This user manual is a supplement to our general wheelchair user manual.

Where necessary, this user manual refers to other manuals as shown below:

 Wheelchair: Refers to the general wheelchair user manual.

Read this user manual and the other user manuals referred to carefully before using the product. If one of the user manuals was not included with your wheelchair, please contact your dealer immediately. In addition to this user manual, there is also a service manual for qualified specialists.

If you are visually impaired, this document can be viewed in PDF format at www.SunriseMedical.eu or alternatively is available on request in large text.

1.2 For further information

Please contact your local, authorised Sunrise Medical dealer if you have any questions regarding the use, maintenance or safety of your wheelchair. In case there is no authorised dealer in your area or you have any questions, contact Sunrise Medical either in writing or by telephone.

For information about product safety notices and product recalls, go to www.SunriseMedical.eu

1.3 Symbols used in this manual

Note!

Pointing out possible problems to the user.

⚠ Caution!

Advice for the user to prevent damage to the product.

⚠ Warning!

Warnings for the user to prevent personal injury.

Not following these instructions may result in physical injury, damage to the product or damage to the environment!

Symbols in illustrations

Each illustration includes icons and numbering to indicate what needs to be done and in which order.

Loosen by one turn with torx

Tighten by one turn with torx

Step 1, 2 etc.

An arrow, as part of an illustration, indicates that the component in question moves or needs to be moved in the direction indicated.

2 Safety

Please note all safety instructions before starting to use the seating system.

Warning!

Always follow the instructions given next to the warning symbols! Not following these instructions may result in physical injury, damage to the seating system, to the wheelchair or to the environment. Where possible, the warnings have been placed in the applicable paragraphs.

2.1 Temperature

If you do not use the seating system, ensure that it is not exposed to direct sunlight for lengthy periods of time. Certain parts of the seating system, such as the seat, the back and the armrests can become hot if they have been exposed to full sunlight for too long. This may cause burns or allergic reactions to the skin.

2.2 Moving parts

Warning!

A seating system and wheelchair have moving and rotating parts. Be careful: contact with moving parts may result in serious physical injury or damage to the seating system or wheelchair.

2.3 General safety guidelines and instructions

Sunrise Medical accepts no liability whatsoever for damage or injury caused by failure to (strictly) adhere to the safety guidelines and instructions or else due to carelessness during the use, service and cleaning of the seating system. Depending on the specific use circumstances, additional safety instructions may be required. The use of the Nuage PLS is being supervised. Please contact your dealer immediately if you observe a potential danger during the use of the product.

Warning!

Open flames, such as lighted matches, can ignite the seating system, leading to serious injury and possible death to the occupant and damage to the seating system, if the occupant is not supervised.

Warning!

Only use the seating system as intended and as prescribed in this manual. Do not adjust or change the product if not promptly described. Do not use parts or add-ons which are not clearly marked as compatible with Nuage seating systems.

To avoid accidents and other problems, it is very important to pay attention to the safety instructions as described in the user manual of the manual wheelchair which the seating system will be used with.

In case of possible technical malfunctioning, please contact your dealer.

3 Product description

Nuage PLS series seating systems are individually adjustable high comfort wheelchair seating solutions for people with limited mobility, trunk balance and postural control. The softly padded seating system 'embraces' the body contours for high comfort, stability, pain and pressure relief.

The seating systems are compatible with most common tilt in space wheelchair frames in the market. Contact your dealer for detailed information and installation.

⚠ Caution!

Please study this manual carefully before using the seating system.

3.1 Nuage PLS series

The Nuage PLS series features:

Nuage PLS (figure 1)

A seating support for full body and leg support of passive users: people with strongly reduced trunk balance and a very limited ability for active sitting.




Nuage PLS Active (figure 2)

A seating support for semi-passive users: people with reduced rump balance and postural stability. The seating system combines body support and a stimulation of (postural) activity.

Nuage PLS Active XS (figure 3)

A seating support for semi-passive children or small adults with reduced rump balance and postural stability. The seating system combines body support and a stimulation of (postural) activity.

For all models a variant with integrated abduction cushion is also available (figure 4).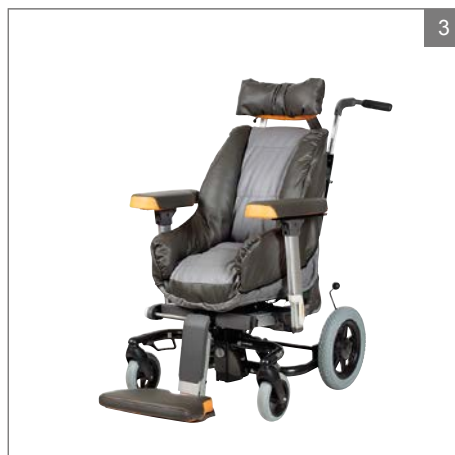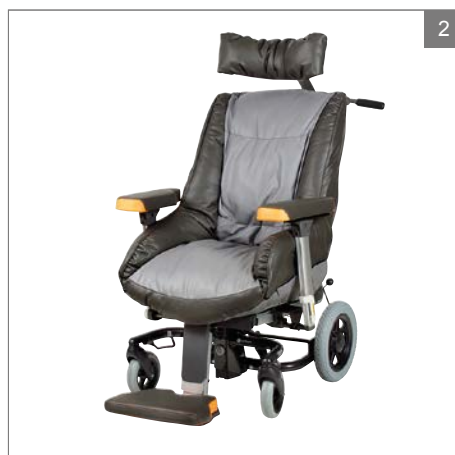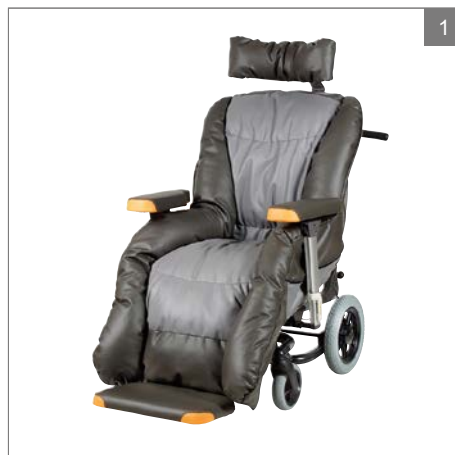


3.2 Positioning options and accessories

The seating systems offer a number of adjustment options to optimize the positioning and support for each individual user.

With the adjustable webbing straps, the seat and backrest contours can be accurately fitted to the body contours of the user. The seating systems also provide asymmetrical positioning options, for example useful in case of pelvic obliquity.

Optional accessories are available for a comfortable support of head, arms and legs. See chapter 11.

The seating system and wheelchair must be adjusted by a qualified specialist. Incorrect settings may have negative consequences for seating posture.

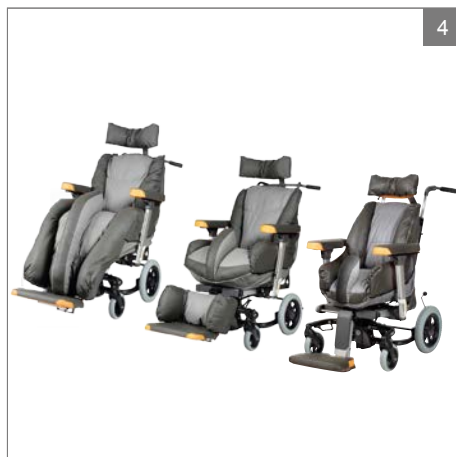4 Use

Nuage PLS seating systems are intended for use on tilt in space wheelchairs. The tilt enables to easily change position. This reduces prolonged pressure on body parts and is therefore important for your health and comfort.

4.1 Getting into and out of the seat

Make sure to engage the wheelchair brakes before getting a person into or out of the seat. Do not stand on the footplates.

⚠ Warning!

Putting your full weight on the footplates may lead to wheelchair damage and to hazardous situations, including the wheelchair tipping up.

4.2 Reducing the seating system in size for transportation

The way a seating system can be reduced in size for transportation without a passenger highly depends on the wheelchair frame it is mounted on. Head rests, armrests and legrests are usually easily removable (📖 Wheelchair). Complete disassembly and reassembly is reserved to professionals, contact your dealer.

⚠ Warning!

Store away detached components safely so that they can not cause injury or damage, or become damaged themselves.

4.3 Use as a passenger seat in a vehicle

Before transporting people in a car or other vehicle as a passenger in a wheelchair with a Nuage PLS seating system, it is recommended to check that compliance with all safety regulations can be assured. Contact your dealer for information and an assessment of your specific situation. Professionals can contact Sunrise Medical for all information on ISO 7176-19 test results.

5

⚠ Warning!

Please check the proper instructions on transportation (📖 Wheelchair)

A wheelchair is not primarily designed to be used as seat in a motor vehicle during transport. Transfer to a regular car seat if possible.

Contact your dealer to check if your wheelchair has been crash tested according to ISO 7176-19 (2008) in combination with your Nuage PLS seating system.

A seating system should be inspected by a Sunrise Medical's representative before reuse after an involvement in any type of vehicle collision.

4.4 Adjusting the seating system

The adjustment possibilities and the way of adjusting depend on the wheelchair frame the seating system is mounted on.

Note!

See information about wheelchair specific adjustment options (📖 Wheelchair)

5 Settings (by a professional)

Seating system adjustments may vary per brand and model of the wheelchair frame. The wheelchair specific adjustments are marked with the addition '(wheelchair specific)'. In these cases, the instructions and figures refer to adjustments of a Nuage PLS on Ibis wheelchairs with Sedeo Pro seating system.

Note!

When you do have another seating system then Sedeo Pro check the manual (📖 Wheelchair) or contact your dealer

Under the cover (A) of a Nuage PLS seating system you find a seat and backrest with straps providing multiple adjustment possibilities for comfortable positioning.

There are two versions of the Nuage PLS seating system with different webbing settings: Nuage PLS I or the more recent version Nuage PLS II. Settings as shown are for Nuage PLS II.

For settings specific to Nuage PLS I see at the end of this chapter.

Note!



For all settings marked with (II) see Nuage PLS I specific settings marked with (I) at the end of this chapter.

5.1 Webbing straps and contour cushion (II)

The backrest contour is formed with 7 webbing straps (B).

The seat contour is formed with a contour cushion (C) in the front and 4 webbing straps (B) in the back. (C) is held in place with 3 additional webbing straps (B).

The webbing straps (B) have orange velcro ends.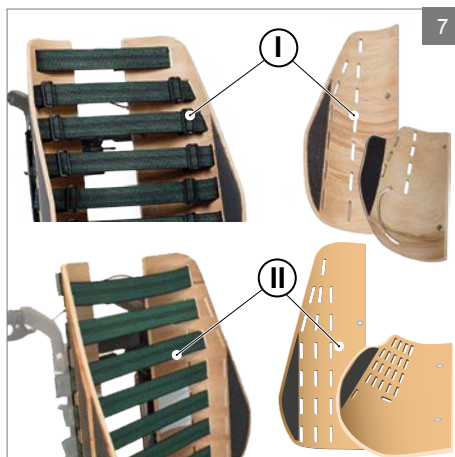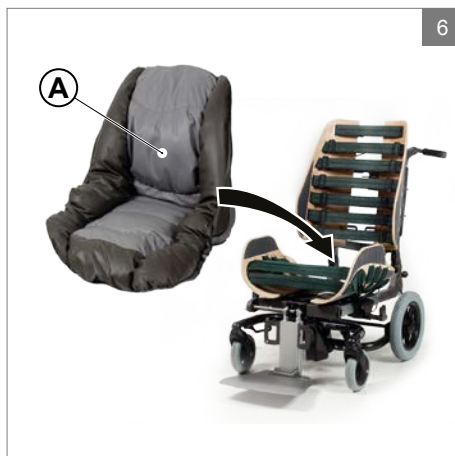

5.2 Seat and backrest shells (II)


The straps are looped through slotted holes in adjustable plywood seat shells (D) and backrest shells (E).

For adjustment purposes seat shell (D) offers 5 rows of slotted holes. Backrest shell (E) offers 3 rows of slotted holes.

Choose which set of holes is best used to accommodate the user. The middle rows marked in black suit the average user.

Warning!

Choose the slotted holes carefully to prevent bottoming-out.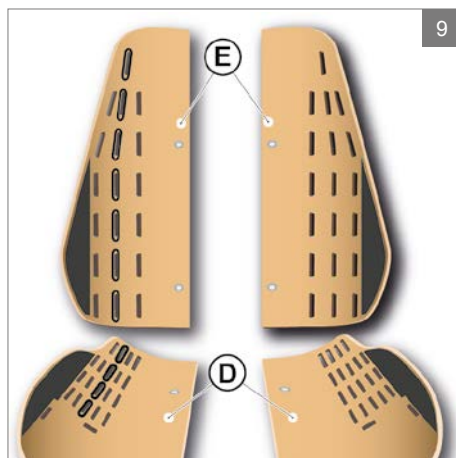


5.3 Seat webbing (II)

The webbing straps (B) enable to follow the contours of the buttocks and provide support where needed for an optimal seating position and stability.

1. Identify the set of slotted holes best suited to accommodate the user (positions F1 to F5).
2. On left and right seat shell (D) loop each strap (B) through a slotted hole (F) or if required over the edge (G).
3. Attach the velcro strap ends to the velcro outer shell surface.
4. Put tension on the webbing straps by stretching them to about 110% of their length.

Warning!


Always check for bottoming-out. If needed change strap position and/or strap tension.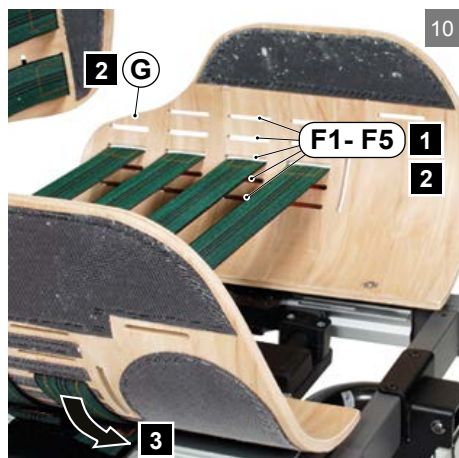

5.4 Contour cushion (II)

The contour cushion (C) helps to create an posterior well that offers postural stability and prevents from sliding forward. Contour cushion (C) consists of 2 side segments, 1 to 3 middle segments and a velcro strap.

1. Determine the number of middle segments to use:
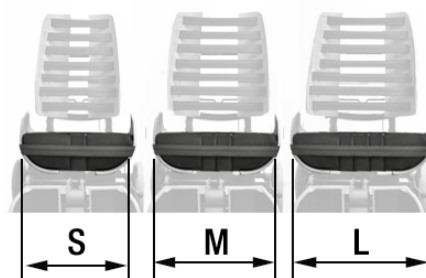


	S	M	L
# Middle segments	1	2	3
Seat width	30-40 cm	40-50 cm	> 50 cm

2. Attach the velcro strap to the bottom side of the cushion segments to hold them together.
3. Loop a webbing strap (B) through slotted holes (H) and attach on the outside of the seat shells with velcro.
4. Place the contour cushion on the seat, back side of the cushion against (B).
5. Loop a webbing strap (B) over the top of the cushion and through slotted holes (I). Attach the strap on the outside of the seat shells with velcro.
6. Place a webbing strap (B) against the front of the contour cushion and attach directly on the outside of the seat shells with velcro.

Caution!

Do not put much tension on the straps (B) that hold the contour cushion in place.

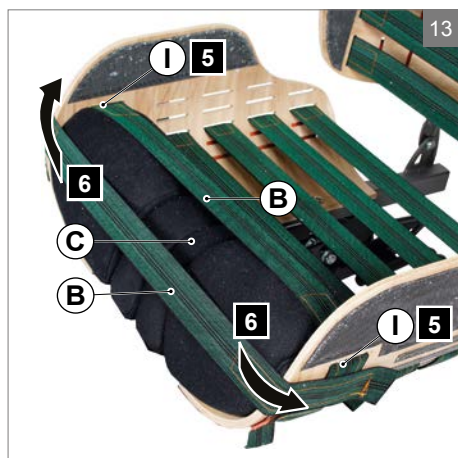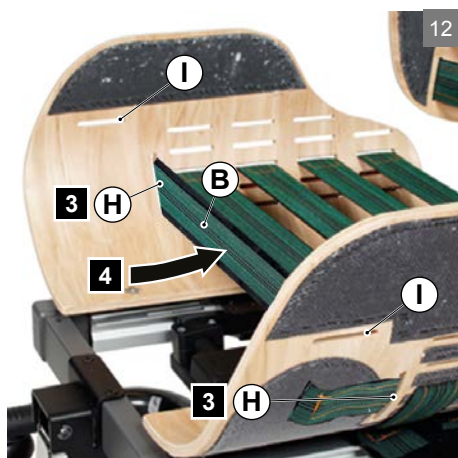5.5 Pelvic obliquity (II)

1. Create an asymmetrical support by choosing different slotted holes (F1 to F5) left and right.
2. Tension the webbing straps (B) in slotted holes (F1 to F5) or if required over edge (G).

5.6 Leg length discrepancy (II)

1. Move one of the seat shells (D1) forwards. To do so, loosen the armrestholder and webbing straps (B), slide it in the rail towards the front of the seat and fasten again.
2. On (D1) shift the position of the webbing straps (B) one slot position to the back. Tension the webbing straps (B) in slotted holes (F1 to F5) or if required over edge (G).
3. Reposition (C) and the front webbings straps (B).

Note!


Step 1 is not possible in combination with seating systems other than Sedeo Pro / Sedeo Original.

5.7 Backrest webbing (II)

The webbing straps (B) enable to follow the contours of the back, create a lumbar support and give further extra support where needed.

1. Identify the set of slotted holes best suited to accommodate the user (positions J1 to J3).
2. On left and right backrest shell (E) loop each strap (B) through a slotted hole (J) or if required over the edge (G).
3. Attach the velcro strap ends to the velcro outer shell surface.
4. Put tension on the webbing straps by stretching them to about 110% of their length.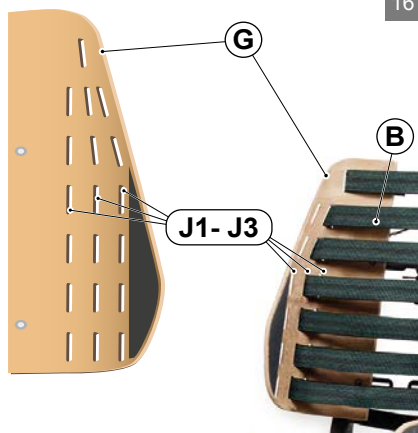

Warning!

Always check for bottoming-out. If needed change strap position and/or strap tension.


5.8 Dorsal obliquity (II)

1. Create an asymmetrical support by choosing different slotted holes (J1 to J3) left and right.
2. Tension the webbing straps (B) in slotted holes (J1 to J3) or if required over edge (G).

5.9 Tensioning the cover

1. Free the cover back flap (O) by detaching the velcro ends (P) at the bottom of the flap from the velcro surfaces on the seat shells.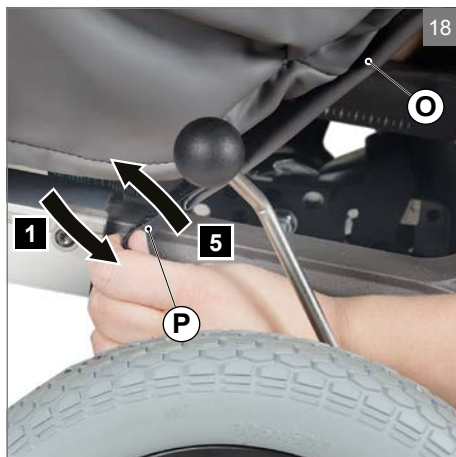


2. Untie the elastic rope (Q) in hem of the cover.
3. Position the cover.
4. Tension the elastic rope (Q) and tie a knot.
5. Close the back flap (O) by attaching the velcro ends (P) to the velcro surfaces on the seat shells.

5.10 Cushion filling

1. Remove the cover (A).
2. Open the zipper (R).
3. Add or remove filling (S).
4. Close the zipper (R).
5. Place the cover (A).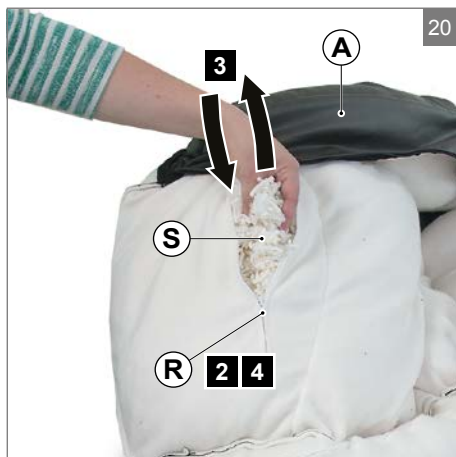


5.11 Seat width (wheelchair specific)

The distance between the seat shells edges (T) and the required seat width relate as
 $(T) = (U) + 12 \text{ cm}$.

Rough adjustment


When mounting the seat, choose the screw hole pattern that fits the required seat width range: 30 to 40 cm, 40 to 50 cm or over 50 cm.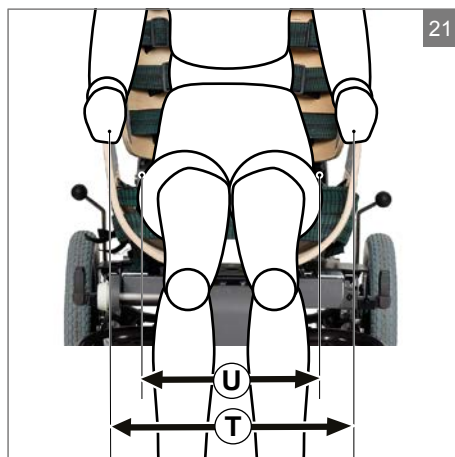

Finetuning

Further adjustment of the seat width for a specific user takes place during a fitting session.

1. Remove the straps on one side by detaching the velcro ends from the velcro surfaces on the seat shells.
2. Loosen (V) on both sides.
3. Move the armrest holder (W) inwards or outwards on both sides.
4. Fasten (V) on both sides.
5. Tension the straps and attach the velcro ends to the velcro surfaces of the seat shells.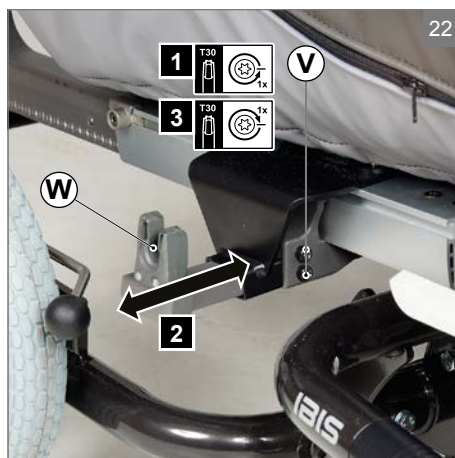

5.12 Seat depth (wheelchair specific)


1. Loosen (X) on both sides.
2. Slide (Y) to the front or back to create the required seat depth.
3. Fasten (X) on both sides.

5.13 Backrest width


The seat width and backrest width are separately adjustable, enabling to position people with broad hips / small shoulders as well as people with small hips / broad shoulders.

1. If required loosen the straps by detaching the velcro ends on one end from the velcro surfaces of the shells.
2. Loosen bolts (A) half a turn with a ring key 13.
3. Slide the backrest shells (B) outwards or inwards.
4. Fasten bolts (A) again.
5. Tension the straps and attach velcro ends on the velcro surfaces of the shells.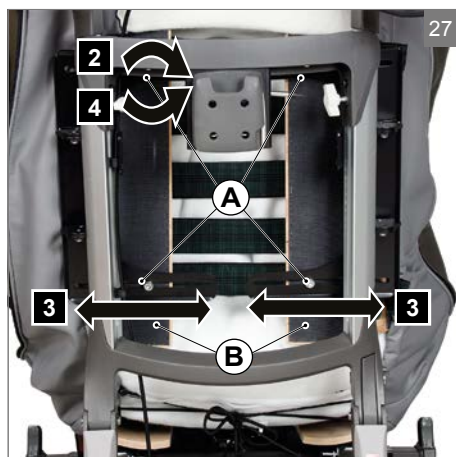


5.14 Backrest height (wheelchair specific)

1. Detach the velcro ends of the cover from the seat.
2. Loosen socket screws (C) .
3. Slide backrest shells (B) up or down.
4. Fasten socket screws (C).
5. Position the cover and fasten cover with the velcro ends.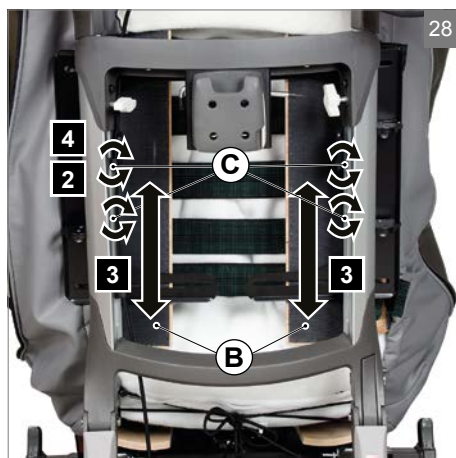

5.15 Funnel shape backrest shells (wheelchair specific)

1. If required loosen straps (D).
2. Loosen bolts (C).
3. Slide the top of the backrest shells (B) apart / closer to each other for more / less shoulder room.
4. Fasten bolts (C).
5. Tension straps (D).
6. Position the cover, tie elastic rope and attach the velcro ends to the seat.

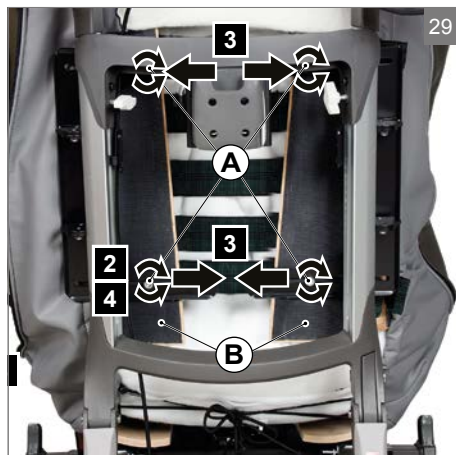5.16 Angle backrest shells (wheelchair specific)

1. If required loosen straps (D) by detaching the velcro ends on one end from the velcro surfaces of the shells.
2. Loosen screws for shell angle adjustment (E).
3. Turn backrest shells (B) inwards or outwards.
4. Fasten (E) again.
5. Tension straps (D).

5.17 Backrest angle (wheelchair specific)

1. Turn (F) to adjust the inclination angle of the backrest (G).

Note!

Alternatively, the wheelchair can be equipped with an optional gas spring actuated angle adjustment (📖 Wheelchair).

5.18 Webbing straps (I)

The seat contour is formed with 3 simple webbing straps (B) and 4 adjustable webbing straps (Z).

The backrest contour is formed with 1 webbing strap (B) and 6 adjustable webbing straps (Z).

The webbing straps (B) have orange velcro ends.

The adjustable webbing straps (Z) have sliding buckles and double velcro ends (orange and black).

5.19 Seat and backrest shells (I)

The straps are looped through slotted holes in adjustable plywood seat shells (D) and backrest shells (E).

Warning!

Tension the straps sufficiently to prevent bottoming-out.

5.20 Seat webbing (I)

The 4 webbing straps (B) in the front and the 4 adjustable straps (Z) in the back enable to follow the contours of buttocks and upper legs while in the mean time providing support where needed for an optimal seating position and stability.

2. Place the adjustable straps (Z) in slotted holes (F), (G) and / or if required over the edge (H).
3. Tension adjustable straps and fasten the velcro ends.
4. Tension further with the sliding buckles.
5. Place the simple straps (B) in slotted hole (I).
6. Tension to approx. 110% of the length and fasten the velcro ends.


Warning!

Always check for bottoming-out. If needed adjust the strap tension.

5.21 Pelvic obliquity (I)

1. Tension the adjustable straps in slotted holes (F) and (G) or if required in slotted hole (G) and over edge (H).
2. Use the sliding buckles and move the velcro ends to create a different (asymmetrical) support left and right.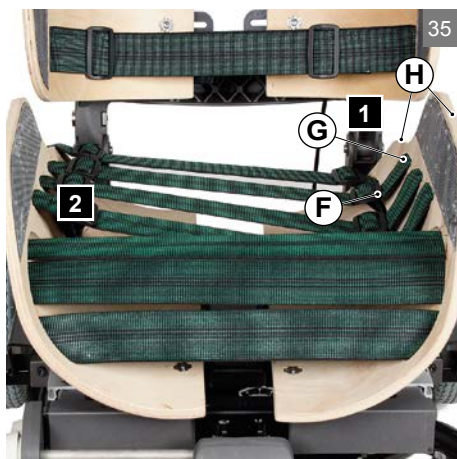

5.22 Leg length discrepancy (I)

1. Move one of the seat shells forwards. To do so, loosen the armrestholder, slide it in the rail towards the front of the seat and fasten again.
2. Tension an adjustable strap from slotted hole (I) to slotted hole (G). At (I) remove the sliding buckle.

Note!

Step 1 is not possible in combination with seating systems other than Sedeo Pro / Sedeo Original.

5.23 Contour cushion (I)

Optionally the front straps can be replaced with a contour cushion.

1. Remove the front webbing straps (B).
2. Put the cushion segments together attaching with the velcro strap to the bottom side and place the contour cushion (C) on the seat.
3. Loop webbing straps (B) over the top of the cushion through slotted hole (I) and attach on the outside of the seat shells with velcro.
4. Place a webbing strap (B) against the front of the contour cushion and attach directly on the outside of the seat shells with velcro.


Caution!

Do not put much tension on the straps (B) that hold the contour cushion in place.

5.24 Backrest webbing (I)

The 6 adjustable straps (Z) and 1 simple webbing strap (B) (Nuage PLS Active XS: five adjustable straps) enable to follow the contours of the back, create a lumbar support and give further extra support where needed.

1. Place a webbing strap (B) in the upper slotted holes (K).
2. Tension strap and fasten the velcro ends.
3. Place adjustable webbing straps (Z) in slotted holes (L), (M) and / or if required over

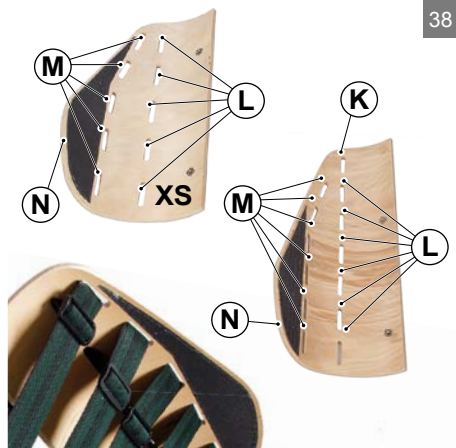the entire edge (N).

4. Tension (Z) and fasten the velcro ends.
5. Tension (Z) further with the sliding buckles.

5.25 Dorsal obliquity (I)

1. Use the sliding buckles and move the velcro ends to create a different (asymmetrical) support left and right.

6 Maintenance

It is recommended that you have your seating system serviced by your dealer once a year or, in the case of intensive use, once every six months.

6.1 Cleaning the upholstery

Cleaning

Clean the upholstery and covers using a damp cloth and household soap. Then rub dry using a soft, dry cloth. In case of persistence stains, allow a few minutes to take effect. Use enzymatic cleaners to remove organic (ureum) stains and odors.

Washing

Follow the washing instructions on the covers. See the washing instructions for the PU-coated covers in figure 40. Before washing remove the padded cushions zipped inside the cover. Allow ample time for drying and make sure the cover is thoroughly dried before use. See the washing instructions for the bamboo terry covers in figure 41.

Disinfecting

Disinfection is a two steps procedure: cleaning (as described above) and decontaminating. To decontaminate PU-coated and bamboo terry covers, use a damp cloth and disinfecting soap. Rinse thoroughly and rub dry using a soft, dry cloth.

⚠ Caution!

Do not use organic solvents such as thinner, benzine or white spirit.

Do not iron or bleach the PU-coated covers.

Do not chemically clean the bamboo terry cover.

6.2 Replacing the upholstery

Replacing or spare PU-coated covers and bamboo terry covers are commercially available.

Polyester knitted
fabric with
polyurethane layer

KEI-
ZER

40

80% Bamboo
20% PES

KEI-
ZER

41

7 Troubleshooting

If there is a problem with your seating system, please contact your dealer.

8 Technical specifications

8.1 CE declaration and requirements

This product complies with the COUNCIL DIRECTIVE 93/42/EEC of 14 June 1993 concerning medical devices. The product also meets the following requirements and standards. This has been verified by independent test organizations.

Standard	Definition/description
EU guideline 93/42 EEC	Requirements as stated in Appendix 1 apply
EN 12183 (2009)	Manual wheelchairs - Requirements and test methods
EN 12184 (2009)	Electrically powered wheelchairs, scooters and their chargers - Requirements and test methods
ISO 7176-19 (2008)	Wheeled mobility devices for use as seats in motor vehicles - Requirements and test methods (commonly referred to as 'crash test')

8.2 Technical information

Nuage PLS seating system and Nuage PLS Active seating system

Description	Unit	
Maximum user weight	kg	120
Seat width	mm	380 - 550
Seat depth	mm	400 - 550
Backrest height	mm	550
Armrest width	mm	120 / 160
Footplate width	mm	410
Footplate depth	mm	360
Calf plate depth	mm	150
Calf plate height	mm	200
Integrated abduction cushion height	mm	140

*Specifications may be influenced by the wheelchair configuration and settings.

**Other specifications are fully wheelchair dependent.

Nuage PLS Active XS seating system

Description	Unit	
Maximum user weight	kg	80
Seat width	mm	250 - 380
Seat depth	mm	250 - 450
Backrest height	mm	450
Armrest width	mm	120 / 160
Footplate width	mm	350
Footplate depth	mm	250
Calf plate depth	mm	150
Calf plate height	mm	200
Integrated abduction cushion height	mm	140

*Specifications may be influenced by the wheelchair configuration and settings.

**Other specifications are fully wheelchair dependent.

9 Warranty

9.1 Definitions of terms

Definitions of terms used in this warranty:

- After sales service part: Part purchased after the initial product that is durable and may be subjected to natural wear and tear or natural contamination during normal operation within the lifetime of the product.;
- Consumable part: Part that is subjected to natural wear and tear or natural contamination during normal operation within the lifetime of the product (section 9 of Sunrise Medical HCM B.V.'s general terms and conditions of sale);
- Client: Those who purchase the product directly from Sunrise Medical;
- Corrective action: Repair, replace or refund of the product;
- Dealer: Those who re-sell the product to the User;
- Defect: Any circumstance due to which the product is not sound or fit to use, caused by a lack of quality of the material used to manufacture the product as well as the quality of the manufacturing process;
- Option: An accessory delivered with the initial product by Sunrise Medical to extend the standard product model;
- Product: Product that is delivered according to brochure or contract (e.g. wheelchair, scooter, battery-charger etc.);
- Part: Part of product that can be exchanged or replaced. This can be an option, accessory, service part or consumable part;
- Returns: Product or part that needs to be returned;
- RMA-process user: Process to return goods, contact your dealer;
- RMA-process client: Process to return goods, contact Sunrise Medical's Customer Service;
- User: Those who use the product;
- Warranty: The rights and obligations set forth in this document;
- Warranty period: The period of time during which the warranty is valid;
- Warranty provider: Sunrise Medical HCM B.V., Vossenbeemd 104, 5705 CL Helmond, The Netherlands (also referred to as Sunrise Medical).

Notwithstanding the rights and obligations of Sunrise Medical, Client and User set forth in Sunrise Medical's general terms and conditions of sale, the rights of the Client and/or User towards Sunrise Medical in case of defects are limited to the provisions set forth in this warranty. For the duration of the warranty period Sunrise Medical guarantees that the product is without defects.

In case of any defects the User is required –within two weeks after discovery of the defect- to contact the dealer. He has to complete a return form and return the product or part via the RMA-process. Sunrise Medical will, at its sole discretion, take the corrective action it seems fit under the given circumstances within a reasonable period of time (depends on nature of claim) from receipt of the completed return form. The warranty period will not be extended after a corrective action.

9.2 Warranty period table

Seating

Description	Warranty period	Examples include, but are not limited to the parts mentioned below
Frame	2 years	Weldment/frame
Electronics	1 year	Electronic components
After sales service parts	New: 1 year after invoice Repaired: 90 days after invoice	Metal parts
Consumable parts	40 days after invoice	Upholstery etc.
Options/ Accessories	2 years	Lap strap, bag brackets etc. Delivered with the initial product

Sunrise Medical will only accept shipment costs and corrective costs related to warranty on equipment during the warranty period.

This warranty will void in case of:

- The product and/or its parts being modified;
- Changes in cosmetic appearance by use;
- Failure to observe the instructions for use and maintenance, use other than normal use, wear and tear, negligence, collateral damage by neglect of earlier symptoms, overloading, third-party accidents, non-original parts used and defects not caused by the product;
- Circumstances beyond our control (flood, fire, etc.).

This warranty does not cover:

- Tyres and inner tubes
- Batteries (covered by the battery manufacturer's warranty).

Clients and/or Users have legal (statutory) rights under applicable national laws relating to the sale of consumer products. This warranty does not affect statutory rights you may have nor those rights that cannot be excluded or limited, nor rights against the entity from whom the product was purchased. Clients may assert any rights they have at their sole discretion.

10 Labelling


10.1 Markings on the seating system

For markings used on the seating system, see the figures.

Note!

Never remove or cover up the markings, symbols and instructions affixed to the seating system. These safety features must remain present and clearly legible throughout the entire lifespan of the seating system.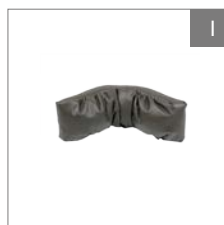

11 Accessories

- A. Contour cushion, Nuage PLS / Nuage PLS Active or Contour cushion, Nuage PLS Active XS
- B. Extra middle segment contour cushion, Nuage PLS, Nuage PLS Active
- C. Footplate, Nuage PLS series
- D. Footplate small, Nuage PLS series
- E. P-shaped armrests pad 16 cm wide, set, Nuage PLS series
- F. P-shaped armrests pad 12 cm wide, set, Nuage PLS series
- G. Flexible armrests pad Platilon, set, Nuage PLS series
- H. Drop-shaped head roll cushion, Nuage PLS series
- I. Headrest, Nuage PLS series or Headrest small, Nuage PLS series
- J. Central calf support, Nuage PLS Active or Central calf support, Nuage PLS Active XS

See our website or contact your Sunrise Medical dealer for more accessories and spare parts.

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KEI-
ZER

Nuage seating systems are products of Revalidatiestoffeerderij Keizer B.V. and are in conformity with the provisions of the medical devices directive concerning class I medical devices.

Sunrise Medical S.r.l.
Via Riva, 20 – Montale
29122 Piacenza
Italia
Tel.: +39 0523 573111
Fax: +39 0523 570060
www.SunriseMedical.it

Sunrise Medical AG
Erlenauweg 17
CH-3110 Münsingen
Schweiz/Suisse/Svizzera
Fon +41 (0)31 958 3838
Fax +41 (0)31 958 3848
www.SunriseMedical.ch

Sunrise Medical AS
Delitoppen 3
1540 Vestby
Norge
Telefon: +47 66 96 38 00
post@sunrisemedical.no
www.SunriseMedical.no

Sunrise Medical AB
Neongatan 5
431 53 Mölndal
Sweden
Tel.: +46 (0)31 748 37 00
post@sunrisemedical.se
www.SunriseMedical.se

MEDICCO s.r.o.
H – Park, Heršpická 1013/11d,
625 00 Brno
Czech Republic
Tel.: (+420) 547 250 955
Fax: (+420) 547 250 956
www.medicco.cz
info@medicco.cz
Bezplatná linka 800 900 809

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Sunrise Medical Australia
11 Daniel Street
Wetherill Park NSW 2164
Australia
Ph: +61 2 9678 6600
Email: enquiries@sunrisemedical.com.au
www.SunriseMedical.com.au

Sunrise Medical
North American Headquarters
2842 Business Park Avenue
Fresno, CA, 93727, USA
(800) 333-4000
(800) 300-7502
www.SunriseMedical.com

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Sunrise Medical B.V.
Sunrise Medical S.A.S