



Technical / News Bulletin

UK

BULLETIN NO.:

TB UK075

DATE ISSUED

11/09/2017

BULLETIN DETAILS

Product Group:	Salsa		
Models Affected:	Salsa M ² & Salsa M ² Mini,		
Effective Date:	14 th August 2017	Serial No. Effective From:	N/A
Bulletin Subject:	Salsa M ² & M ² Mini Swing Away Mount Update		

Description:

As part of our ongoing post market surveillance, it has come to our attention that the swing away mount on the Salsa M² and Salsa M² Mini, has become loose over time on a number of chairs in the field.

The design of the swing away mount has been improved to overcome this challenge.



Old swing away mount

Corrective Action

We have now added a washer and friction pad. This means that the bolt is now fastened against a non-moving part. This ensures that the bolt does not move when the swing away mount is used and in turn will not become loose.



New swing away mount

The new swing away mount solution is effective on chairs built from **Monday 14th August 2017**

For chairs currently in the field this new solution can be fitted by contacting help.technical@sunmed.co.uk and ordering part number **21504015**. These will be supplied free of charge for chairs in the field.

Literature Affected:

Website:	N.A
Order forms:	N.A
Owners Manual:	Available on www.sunrisemedical.co.uk

If you have any questions regarding this bulletin, please do not hesitate to contact:

Customer Services	Customer Support	Technical Service Centre	E-mail
Telephone:	+44 (0)845 605 66 88 Option 2	+44 (0)845 605 66 88 Option 3	enquiries@sunmed.co.uk
Fax:	+44 (0)845 605 66 89	+44 (0)845 605 66 89	help.technical@sunmed.co.uk