



Technical / News Bulletin

UK

BULLETIN NO.:

TB 0542

DATE ISSUED

21.05.2018

BULLETIN DETAILS

Product Group:	QUICKIE		
Models Affected:	Jive Up & Jive M2 Sedeo Ergo		
Effective Date:	4 th June 2018	Serial No. Effective From:	
Bulletin Subject:	Quickie Jive Up & Jive M2 Sedeo Ergo Discontinuation		

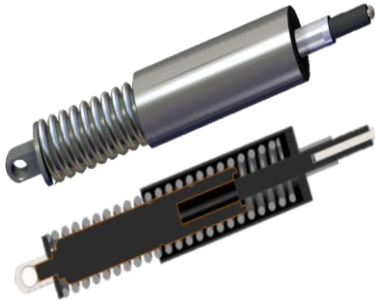

Description:

With the recent launch of the NEW Quickie Q700 Up M and the forthcoming Quickie Q700 Sedeo Ergo M, we will be discontinuing the Jive Up and Jive M2 Sedeo Ergo as of the 4th June 2018. This is due to the latest versions effectively making the latter obsolete for clients.

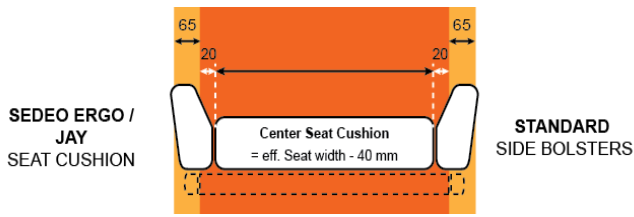
Last orders to be processed for both the Jive Up and Jive M2 Sedeo Ergo will be on the 4th June 2018.

Benefits to Customer

The NEW Quickie Q700 Up M and Quickie Q700 Sedeo Ergo M bring significant improvements and benefits to both the client and dealers.

Feature	<u>Benefit</u>
<p>NEW Spider Trac 2.0 Suspension – The NEW Spider Trac suspension has split the function of the gas strut into two physical parts. Firstly the anti-pitch mechanism is driven through the outcoil spring which provides the chair's stability and traction by forcing the castor arms to the ground. Secondly the new gas chamber, which provides the dampening function for descending kerbs, is now atmospheric pressure rather than the previous 400psi. This means that the gas chamber's internal forces are less and provides an increased lifecycle.</p> 	<ul style="list-style-type: none"> • Increased reliability and life expectancy • Retro-fitable to existing Jive M chairs.
<p>NEW Deep Contour Backrest – New Air Flow Deep Contour backrest has been designed based on the JAY Comfort Deep Contour.</p> 	<ul style="list-style-type: none"> • Providing greater lateral stability for clients • Reduces need to use Built 4 Me to adapt for JAY Comfort Deep Contour

NEW Slim line Side Bolsters – Slim side bolsters have been introduced to make it easier to prescribe the correct JAY cushion size without compromising on the overall width of the chair or styling.



- Retains Sedeo Ergo style
- No need to compromise on JAY cushion size verses overall width of the chair
- Easier to prescribe, as when the slim line side bolsters are selected, the centre cushion is the seat width of the client

NEW Headrest Stems & Pads – Easily prescribe multiple headrests with the new robust and compact Sedeo stem or Whitmyer Linix stem.

Also we have a range of Whitmyer headrests on the order forms which make it easier to match the correct headrest pad to the client's clinical needs and comfort requirements (**Sedeo Pad Small, Sedeo Pad Small with Cover, Sedeo Pad Flexible, Whitmyer Plus Pad & Whitmyer Adjust A-Plush**).



- Simpler to prescribe Whitmyer
- Increased choice for the client to make it easier to match the headrest pad to the client's comfort and clinical needs
- Reduced need to go off the prescription form to fit third party headrests

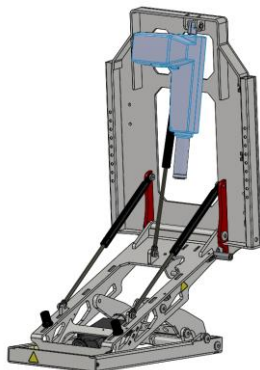
Improved Styling – Six universal colour clips (White, orange, green, blue, black & red)






- More customisable for the customer

NEW Double Tilt Module –

We have a new double tilt module which has a number of improvements. These are focused on reducing the play of the module while in stand, firstly we have introduced a new third gas spring, secondly increased material thickness and finally reduced the number of joints on the module to six. With the introduction of the new double tilt module we have been able to increase the standing angle from 57° (Jive Up) to 73° allowing the client to be positioned more upright.



- Clients can stand up to 73 degrees, this provides pressure relief for clients who can't independently shift weight, promotes bone health, supports vital organ capacity including pulmonary, bowel and bladder function, improves circulation, enhances client's social interaction and improves the ability to reach and access work and home environment
- Due to the stiffer module while standing, the customer has increased confidence in the chair and feels safer

<p>NEW 90° legrest angle (Q700 UP M)</p> 	<ul style="list-style-type: none"> • Easier for clients to get closer to work surfaces • Simpler to position clients with tight hamstrings in the Q700 Up comfortably
<p>NEW 180mm backrest recline – Minimises the effects of shear forces through the entire backrest range of travel.</p> 	<ul style="list-style-type: none"> • Prevents skin breakdown when operating the backrest recline actuator function • Provides the most consistent positioning for both head or chin controls through the entire range of movement of the recline
<p>Reclining Armrests standard option</p>	<ul style="list-style-type: none"> • Provides the optimal positioning of the armrests in any position as the armrests follow the positioning of the backrest
<p>Six Memory Positions – Personalise the Q700 UP or Sedeo Ergo for the client by programming commonly used positions such as lay flat, stand, transfer, watching TV, seated or driving outside. This makes it easy and quick for the client to access commonly used positions and each of these can be tailored exactly for the client.</p> 	<ul style="list-style-type: none"> • Simple to access commonly used positions via the touch of one button using QUICKIE's patented assignable buttons function • Encourages better positioning/postural management than individual actuator movements, as multiple actuators work in conjunction of each other to reposition the customer to the desired position i.e. for lay flat to seat the actuators that are used are foot lift, foot extend, backrest recline, these actuators work in sync so that the final position doesn't leave the client slumped forward but rather in the back of the seat.
<p>NEW Firmware – The Q700 Up M provides a smooth and seamless transition while entering into memory positions</p>	<ul style="list-style-type: none"> • Provides a smoother and more natural transition into memory positions
<p>Advanced Tailored Programming – With the Q700 Up M & Q700 Sedeo Ergo M it is possible to tailor not just the end stop of the memory position, but how the chair enters into memory positions. This is achieved through advanced programming which is available in the Q700 Up M & Sedeo Ergo M. i.e. seated – lay flat – stand or seated – pressure relief stand.</p>	<ul style="list-style-type: none"> • Simpler and more comfortable for the user to enter a memory position • Provides the ability to tailor the drive setting and the seating inhibits to the clinical and lifestyle requirements of the user, enabling them to live with fewer compromises in their life.

NEW Link-IT Special Control Mounting System - Link-IT

is our NEW mounting system for special controls (SID's) and switches.

Link-IT mounting kit consists of various lightweight interconnecting pieces, which when combined, are robust and infinitely adjustable. Making it easy to position SID's and switches precisely for the client.



MicroPilot on Bib Mount



MicroGuide with Mode Switches



All Round with Mode Switch



OMNI Mount



MicroGuide in Space Disk



All Round on Chin Boom

- Accurate – allowing you to position them exactly where the user requires
- Quick to mount through the use of one tool and with minions and rods which are infinitely adjustable
- Neat – Compact minions and rods stand out less than traditional mounts

With the launch of the QUICKIE Q700 Up M & up-coming launch of the QUICKIE Q700 Sedeo Ergo M we have identified a selection of QUICKIE dealers already operating in this area of the market, however if you are interested in becoming a QUICKIE Q700 Up M & Q700 Sedeo Ergo M dealer please do not hesitate to contact Samuel Owen (Samuel.owen@sunmed.co.uk)

For more information on the QUICKIE Q700 Up M & QUICKIE Q700 Sedeo Ergo M please visit www.sunrisemedical.co.uk

Literature Affected:

Website:	N.A
Order forms:	N.A
Owner's Manual:	N.A
Parts Manual:	N.A

If you have any questions regarding this bulletin, please do not hesitate to contact:

Customer Services	Customer Support	Technical Service Centre	E-mail
Telephone:	+44 (0)845 605 66 88 Option 2	+44 (0)845 605 66 88 Option 3	enquiries@sunmed.co.uk
Fax:	+44 (0)845 605 66 89	+44 (0)845 605 66 89	help.technical@sunmed.co.uk