

	Technical / News Bulletin UK		BULLETIN NO.:	
			TBUK115	
			DATE ISSUED	
			17/06/2019	
BULLETIN DETAILS				
Product Group:		QUICKIE		
Model Affected:		Q700-M Pro / Sedeo Ergo / UP & JIVE M / Jive M2 Sedeo Ergo / Jive Up		
Effective Date: See Below		Serial No. Effective From:	N/A	
Bulletin Subject:		QUICKIE Q700-M / Jive-M Update		
Description of Issue / Solution:				
<p>We introduced the Q700-M since March 2018 with this introduction we began to offer our NEW Spider Trac 3.0 Suspension.</p> <p>The development and aim of the Spider Trac 3.0 suspension on the Q700-M was to offer the same ride performance that users and dealer have experienced over the years with the Jive. Such as the dampened descents down kerbs of up to 5" or the kerb climbing ability of 4" while significantly improving the longevity from the gas strut itself.</p> <p>Since this introduction the Q700-M and the Spider Trac 3.0 have been exceedingly well received from both users and dealers alike. Therefore with immediate effect we will be discontinuing the old Spider Trac suspension parts as the Spider Trac 3.0 can also be retro-fitted on to existing Jives in the field.</p> <p>The NEW kit part numbers are below along with which QUICKIE product they can be used for:</p>				
Product		Suspension Kits – Black Gas Strut & Performance Spring (Pair)	Suspension Kits – Red Gas Strut (Pair)	
		21815101-020	21815102-010	
Jive M		x		
Jive M2 Sedeo Ergo		x		
Jive UP M			x	
Q700-M Pro		x		
Q700-M Sedeo Ergo		x		
Q700-UP M			x	
<p>Step by step instruction videos have been created to assist with fitting the kits to the new chairs. The how-to video link is as followed;</p> <p>Jive M Gas Spring exchange from previous strut to the new version - https://youtu.be/4JHq_JlQhiU</p> <p>Performance Spring exchange Q700 M - https://youtu.be/jcQR_crlGUc</p> <p>Performance Spring exchange Jive M - https://youtu.be/ZTYbNSoUHds</p> <p>Exchange Bronze Block Jive M/Q700 M - https://youtu.be/fCMqgmzLaggE</p> <p>If you have any questions please contact your local Area Sales Manager or our Technical Service Department.</p>				

Customer Services	Customer Support	Technical Service Centre	E-mail
Telephone;;	+44 (0)845 605 66 88 Option 1-1	+44 (0)845 605 66 88 Option 1-2	enquiries@sunmed.co.uk
Fax:	+44 (0)845 605 66 89	+44 (0)845 605 66 89	help.technical@sunmed.co.uk