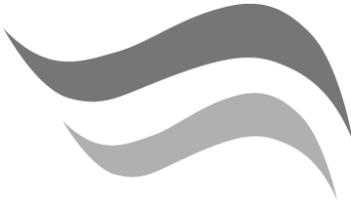
# Entrée

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## Foreword

Dear Customer,

We are very happy that you have decided in favour of a high-quality product from SUNRISE MEDICAL. This user's manual will provide numerous tips and ideas so that your new wheelchair can become a trustworthy and reliable partner in your life.

Maintaining **close links** with our customers is of great importance to us at Sunrise Medical: we therefore like to keep you up-to-date with the latest developments at Sunrise Medical. Keeping close to our customers also means fast service when you need replacement parts or accessories, or just have a question about your wheelchair – and with as little red tape as possible.

We want you to be satisfied with our products and service. Sunrise Medical therefore constantly works to continuously develop its products. For this reason, changes can occur in our range of products with regard to shape, technology and fittings. Consequently, no claims can be construed from the data or pictures contained in this user's manual.

SUNRISE MEDICAL has been awarded the ISO- 9001 Certificate, which affirms the quality of our products at every stage, from R & D to production.

Please contact your local, authorised SUNRISE MEDICAL dealer if you have any questions concerning the use, maintenance, or safety of your wheelchair.

In the case that there is no authorised dealer in your area or you have any questions, you can contact Sunrise Medical either in writing or by telephone at:

**Sunrise Medical GmbH**  
**High Street Wollaston**  
**West Midlands DY8 4PS**  
**England**

Phone +44 (0) 1384 446622

Fax +44 (0) 1384 446644

[www.sunrisemedical.com](http://www.sunrisemedical.com)

## Table of Contents

|                              |    |
|------------------------------|----|
| Foreword .....               | 11 |
| Safety Tips .....            | 12 |
| Wheelchair components .....  | 13 |
| Handling .....               | 14 |
| Options                      |    |
| Step Tubes .....             | 14 |
| Brakes .....                 | 14 |
| Footplate .....              | 15 |
| Castors .....                | 15 |
| Axle Plates .....            | 15 |
| Backrest .....               | 15 |
| Armrest .....                | 16 |
| Lap Belt .....               | 16 |
| Anti-Tip Tubes .....         | 16 |
| Seat .....                   | 16 |
| Crutch Holder .....          | 16 |
| Tyres and Mounting .....     | 17 |
| Possible Problems .....      | 17 |
| Maintenance and Care .....   | 17 |
| Technical Data .....         | 17 |
| Nameplates / Guarantee ..... | 18 |
| Torque .....                 | 18 |

This manual gives information on all features sold across different countries, without stating whether they are actually available in your country or are optional or standard features. For this information please refer to the orderform/prescription form or your prescriber/supplier.

## Safety Tips

The engineering and construction of this wheelchair have been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded. Nevertheless, users may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must absolutely be observed. Unprofessional or erroneous adaptation or adjustment work could increase the risk of accidents. As a wheelchair user, you are also part of the daily traffic on streets and sidewalks, just like anyone else. We would like to remind you that you are thus also subject to any and all traffic laws. For this reason, you should always wear light clothing in the dark as much as possible so that you can be seen more easily and make sure that the reflectors are not covered. Be careful during your first ride in this wheelchair. Get to know your wheelchair. Test what effect changing the center of gravity would have on your wheelchair. Practice using your wheelchair on level ground as well as going uphill and downhill.

### CAUTION:

The wheel locks are not intended to brake your wheelchair. They are only there to ensure that your wheelchair does not begin rolling unintentionally. When you stop on uneven ground, you should always use the wheel locks to prevent such rolling. Always set both wheel locks; otherwise, your wheelchair could tip over.

**The wheel locks have not been designed to be used as brakes for a moving wheelchair.**

Always make use of elevators and ramps. If these are not available, you can reach your destination with the aid of two attendants.

**They should grip the wheelchair at securely mounted parts.** If your wheelchair is equipped with anti-tip tubes, these should be folded away. A wheelchair should never be lifted with an occupant; it should only be pushed.

Before each use, the following should be checked:

- Quick-release axles on the rear wheels
- Velcro on seats and backrests
- Tyres, tyre pressure, and wheel locks.

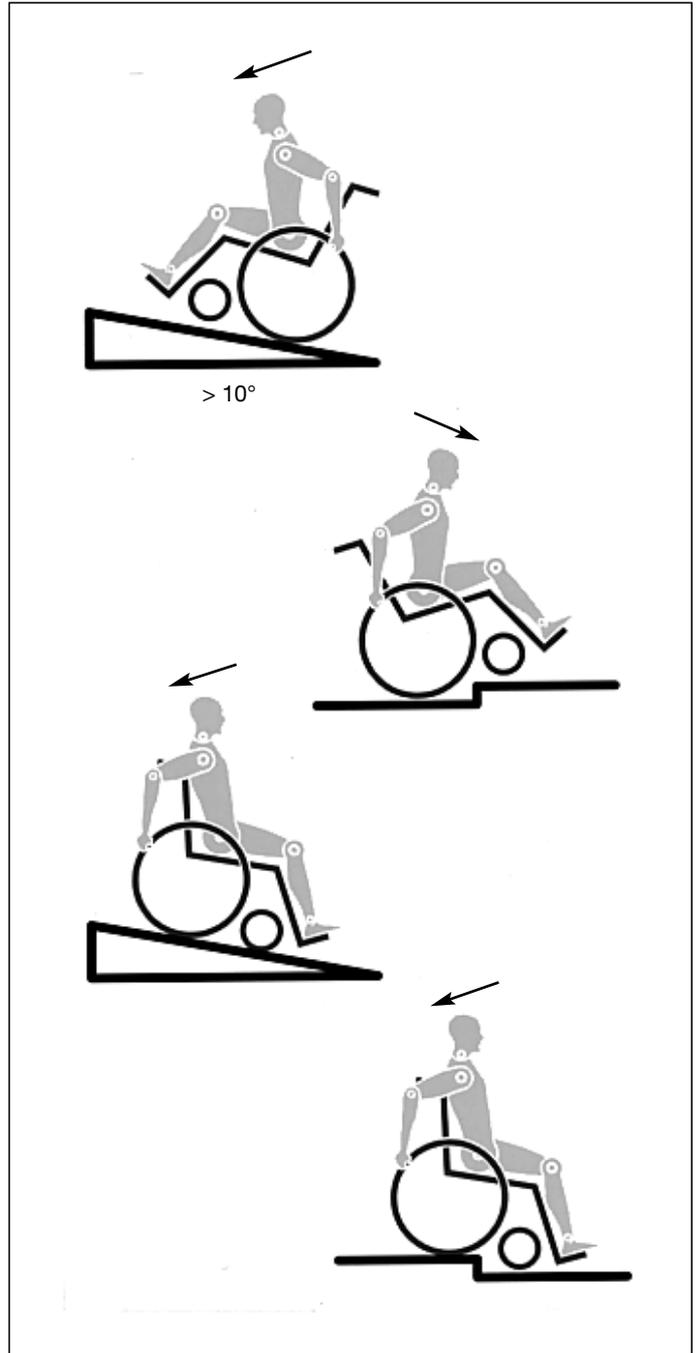
**Wheel locks will not function properly if tyre pressure is too low or the interval between tyres and wheel locks is too large.**

**The maximum user weight is 125 kilograms (20 stone).**

This wheelchair should be used exclusively to convey **one person** at a time in the seat. Any other use beyond this limit is in violation of safety regulations.

**Users should not sit in the wheelchair while riding in any and all means of transportation.**

Only permanently installed seats and seat belts will offer sufficient protection in hazardous situations. When vehicles are in motion, unoccupied wheelchairs should be secured using appropriate means. Before changing any of the adjustments of this wheelchair, it is important to read the corresponding section of the user manual. It is possible that potholes or uneven ground could cause this wheelchair to tip over, especially when riding uphill or downhill. When riding over a step or up an incline frontally, the body should be leaning forward.

## Transportation

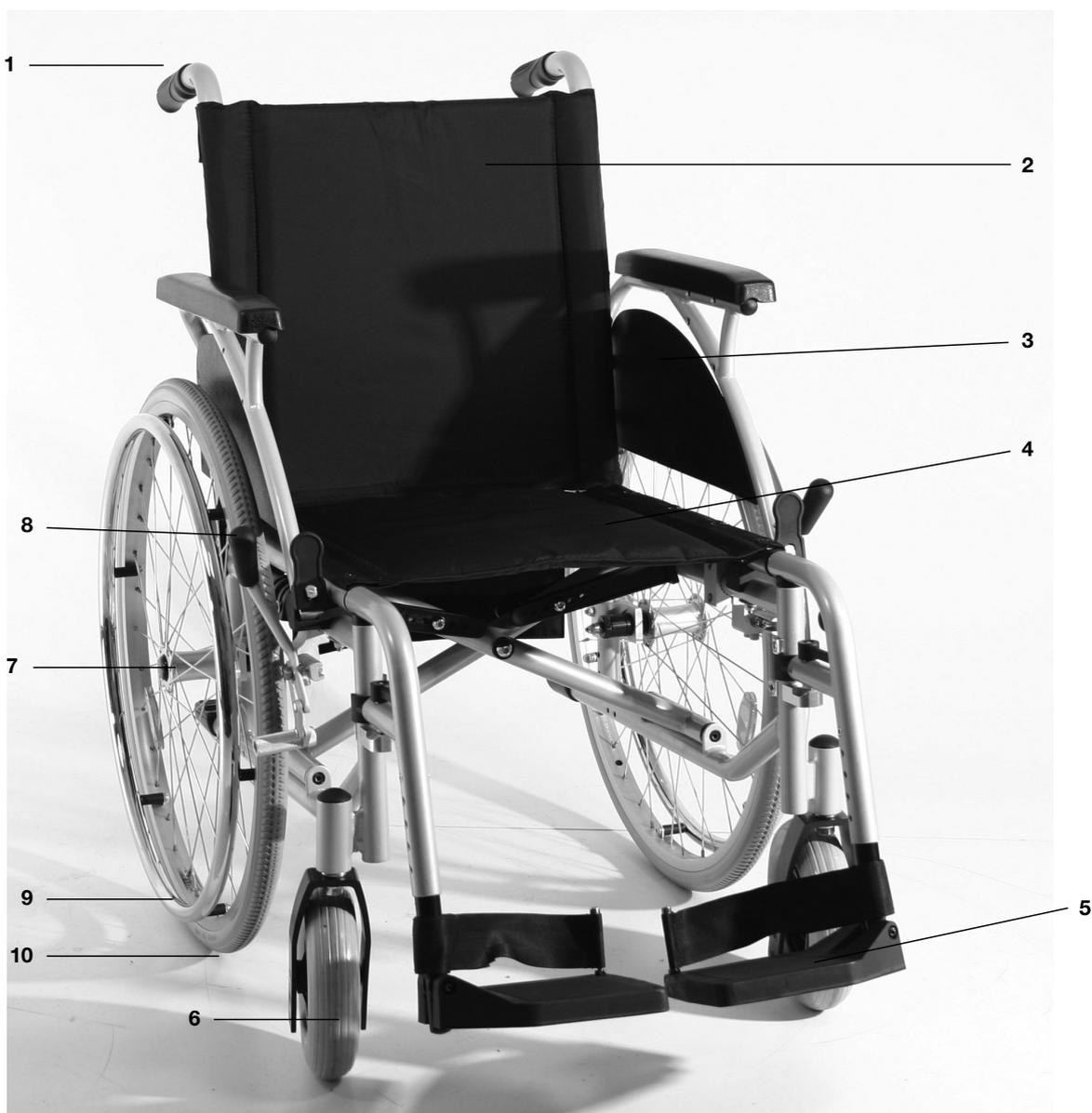
**Your wheelchair is not suitable for use as a seat in vehicles. Do not sit in the wheelchair during transportation. Use the vehicle seat or a properly secured transport approved seat.**

## Wheelchair components

We at SUNRISE MEDICAL have been awarded the ISO 9001 Certificate, which affirms the quality of our products at every stage, from R & D to production. This product meets relevant National and International Standards to satisfy compliance with EU directives. Optional equipment and accessories are available at extra charge.

### Wheelchairs:

1. Push handles
2. Backrest upholstery
3. Sideguards
4. Seat sling
5. Footrest
6. Castors
7. Quick-release axles
8. Wheel locks
9. Handrim
10. Rear wheel

## Handling

### Folding up

First remove the seat cushion and back shell (comfort version only) from the wheelchair and flip up from the wheelchair and flip up from the (platform) footplate or the individual footplates. Take hold of the sling from underneath in the middle and pull upwards. The wheelchair will then fold up. To fold your wheelchair until it is as small as possible, e.g., to put into a car, you can remove the footrests (depending on the model). For this purpose, open the latch from the outside and swing the footrest to the side. Then the footrest only has to be drawn out of the frame tubing.

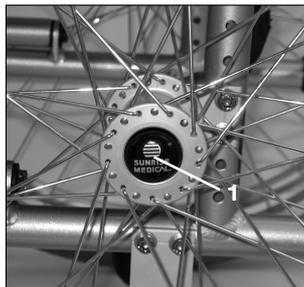
### Unfolding

Press down on the seat tubing (see picture). Your wheelchair is then unfolded. Now snap the seat tubing into position in the seat saddle. This can be facilitated by slightly tipping your wheelchair. In this way, the weight can be taken off of one rear wheel so that the wheelchair can be more easily unfolded. Be careful not to get your fingers caught in the cross-tube assembly. Finally put seat cushion and backshell on again.

*Note: Make sure that the back shell snaps in correctly.*

### Quick-Release Axles for Rear Wheels

The rear wheels are equipped with quick-release axles. The wheels can thus be installed or removed without using tools. To remove a wheel, simply depress the quick-release button on the axle (1) and pull it out.

#### CAUTION:

**Hold the quick-release button on the axle depressed when inserting the axle into the frame to mount the rear wheels. Release the button to lock the wheel in place. The quick-release button should snap**

## Options – Step Tubes

### Step Tubes

Step tubes are used by attendants to tip a wheelchair over an obstacle. Simply step on the tube to push a wheelchair, for example, over a kerb step.

## Options – Brakes

### Wheel Locks

Your wheelchair is equipped with two wheel locks. They are applied directly against the tyres. To engage, press both wheel-lock levers forward against the stops (Fig. 1 and 2). To release the wheel, pull the levers back to their original positions.

Braking power will decrease with:

- Worn tyre tread
- Tyre pressure that is too low
- Wet tyres
- Improperly adjusted wheel locks.

The wheel locks have not been designed to be used as brakes for a moving wheelchair. The wheel locks should therefore never be used to brake a moving wheelchair. Always use the hand-rims for braking. Make sure that the interval between the tyre and wheel locks complies with given specifications (Fig. 1 and 2). To readjust, loosen screw (1) and set the appropriate interval. Tighten screw (see the page on torque)

1

2

#### CAUTION:

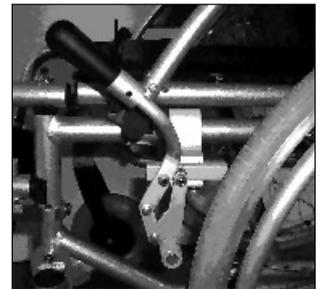
**After each adjustment of the rear wheels, check the interval to the wheel locks and readjust if necessary.**

### Extension for the wheel lock Lever

The longer lever helps to minimise the effort needed to set the wheel locks.

#### CAUTION:

**Mounting the wheel lock too close toward the wheel will result in a higher effort to operate. This might cause the wheel lock extension lever to break!**

Leaning onto the wheel lock extension lever while transferring will cause the lever to break! Splashing water from tyres might cause the wheel lock to malfunction.

### Drum Brakes

Drum brakes permit safe and convenient braking for an attendant. They can also be set with the aid of a locking lever (1) to prevent rolling. This lever must snap audibly into place. Drum brakes are not dependent on the aid of air pressure inside the tires. Your wheelchair cannot be moved when the drum brakes are set.

#### CAUTION:

**Drum brakes should only be adjusted by authorized dealers.**

## Options – Footplate

### Footrest

The footrests can be flipped up to make it easier to get into and out of the chair. By loosening the screw (1) the angle can be adjusted in the area of  $\pm 15^\circ$ . Then re-tighten the screw (see Torque).

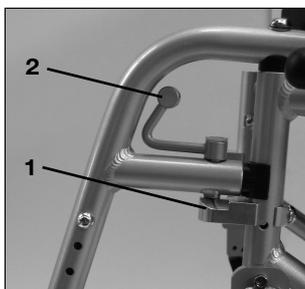
### Lower leg length

By loosening the screw (1) the lower leg length can be adjusted. Loosen the screws and adjust the tube with the footrest to the desired position. Then re-tighten the screw (see page on Torque). Make sure that the gap to floor is not below the minimum of 2.5 cm.

### Legrests and Latch

The legrest can be swung in and outwards. Fit your legrest by inserting it with the footplate(s) pointing toward the outside. Then rotate it toward the inside until it locks into place (1). To remove the legrest, depress the lever (2), swing the footrest towards the outside and lift to remove it. Make sure that the legrest is properly locked into place.

### CAUTION:

The legrests should not be used to lift or to carry the wheelchair.

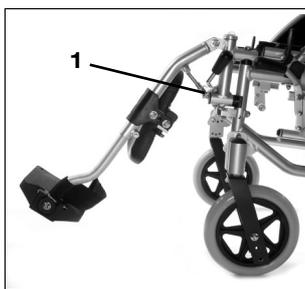
### Elevating legrest

#### To elevate:

Lift your lower leg and press the release button. Pull the legrest upwards and adjust it to the desired height. As soon as you release the button, the legrest will click back into place.

#### To lower:

Lift your lower leg and press the release button to unlock the legrest to lower it (1). As soon as you release the button, the legrest will click back into place.

### CAUTION:

The elevating legrest is not allowed to be adjusted into the lowest position to avoid any interference with the castors.

### CAUTION:

Keep hands clear of the adjustment mechanism between the frame and the movable parts of the legrest while elevating or lowering the legrest.

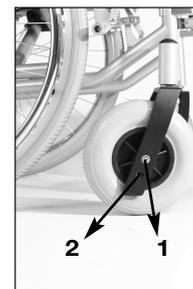
### CAUTION:

Legrests are not to be used for lifting or carrying the wheelchair with an occupant.

## Options – Castor

### Adjustment of seat height and seat angle

The seat height can be adjusted by moving the castor in the fork SH 47 cm (1), SH 51 cm (2).

and the rear wheel.

### ATTENTION:

Adjust the seat height only parallel.

## Options – Seat

### Standard seat sling

The seat sling has Velcro® fastening on each side. This makes it possible to re-tension it easily. To re-tension the seat sling, fold the wheelchair slightly. Remove the screws (1).

By opening the Velcro® fastening the seat sling can now be re-adjusted. To re-assemble, follow the procedure in reverse order. Make sure that after the adjustment work, all screws are re-tightened (see page on Torque).

### ATTENTION!

In order to ensure safety, at least 50% of the Velcro® surface must be in contact.

## Options – Crutch Holder

### Crutch Holder

This device permits crutches to be transported directly on a wheelchair. It has a Velcro loop (1) to fasten crutches or other aids.

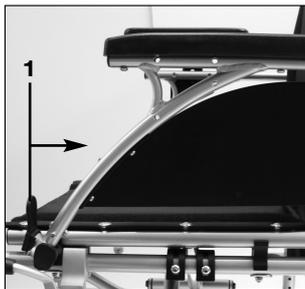
### CAUTION:

Never try to use or even remove the crutches or other aids while moving.

**Options – Armrest**

**Standard sideguard, flip-up, removable with short or long arm pads (7.81.3)**

With the sideguard which is stepped down at the front it is possible to move up closer to a table. To flip them up press the lever (1) forwards, so that the sideguard is released.


**ATTENTION!**

**Sideguards, and the armrests are not designed to be used to lift or carry the wheelchair.**

**Sideguard, flip-up, removable with short or long arm pads, height-adjustable**

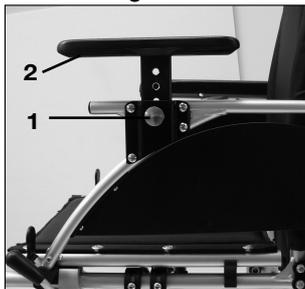The arm pad can be height-adjusted in the following way. Pull the lever (1) and adjust the arm pad (2) to the desired height. Release the lever and push the arm pad (2) down until you hear it click into place.

**ATTENTION!**

**Sideguards, and the armrests are not designed to be used to lift or carry the wheelchair.**

To increase the belt length: - To reduce the belt length: -

|   |  |   |
|---|--|---|
| <p>Feed the belt through slide adjusters and male buckle to provide more belt length.</p> | <p>Feed the belt back through male buckle and slide adjusters.</p> | <p>Ensure the belt is not looped at the male buckle</p> |
|---|--|---|

When fastened check space between the lap belt and the user, when correctly adjusted it should be possible to insert the flat of the hand between the lap belt and the user.

Generally the Lap Belt should be fixed so that the straps sit at an angle of approximately 45°, and when correctly adjusted should not allow user to slip down in the seat.

**Options – Anti-Tip Tubes**

**Anti-tip tubes**

The anti-tip tubes (1) give greater safety to user when they are learning to use the chair. The anti-tip tubes (1) stop the wheelchair tipping over backwards. The anti-tip tubes (1) can be pulled backwards so that they are released and can be flipped up. By pressing the release button, they can be removed. There should always be a gap of 3 to 5 cm to the floor. When travelling up and down a high step (e.g. kerb) the anti-tip tubes must be flipped up in order to avoid them catching on the kerb.

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**Options – Lap belt**

**⚠ Before using your wheelchair ensure the seat belt is worn and correctly adjusted before use.**

The lap belt is fitted to the wheelchair as shown in the illustrations. The lap belt is fitted between the armrest support and the backrest tube. The belt should then be run across the outside of the back upholstery.

**⚠ It is important that the belt is routed around the back of the wheelchair. The buckle end can be fed between the armrest and backrest.**

Adjust the belt position so that the buckles are in the centre of the seat.

**Adjust the lap belt to suit the user's needs as follows:**

|   |  |
|---|--|
| <p><b>To fasten buckle: -</b><br/>Firmly push the male buckle into female buckle.</p> | <p><b>To release belt: -</b><br/>Press the exposed sides of the male buckle and push towards the centre whilst gently pulling apart.</p> |
|---|--|

**⚠ Do not rely on the lap belt only when the wheelchair and occupant are transported in a vehicle, use the separate occupant lap and diagonal restraints provided in the vehicle.**

**Advice to Client**

The lap belt must be checked on a daily basis to ensure it is adjusted correctly and it is free from any obstruction or adverse wear.

**⚠ Failure to make sure that the lap belt is secure and adjusted prior to use could cause serious injury to the user. E.g. too loose a strap may allow the user to slip down in the chair and risk suffocation.**

**Maintenance**

Check lap belt, and securing components; at regular intervals for any sign of fraying or damage. Replace if necessary. Clean the lap belt with warm soapy water and allow to dry.

**Note**

**The lap belt should be adjusted to suit the end user as detailed above**  
Sunrise Medical also recommend that the length and fit of the belt is checked on a regular basis to reduce the risk of the end user inadvertently re-adjusting the belt to an excessive length

**⚠ If in doubt about the use and operation of the seat belt then ask your healthcare professional, wheelchair dealer, carer or attendant for assistance.**

## Tyres and Mounting

### Tyres and Mounting

Always make sure you that you maintain the correct tyre pressure, as this can have an effect on wheelchair performance. If the tyre pressure is too low, rolling resistance will increase, requiring more effort to move the chair forward. Low tyre pressure also has a negative impact on maneuverability. If the tyre pressure is too high, the tyre could burst.

The correct pressure for a given tyre is printed on the surface of the tyre itself.

Tyres can be mounted the same way as an ordinary bicycle tyre. Before installing a new inner tube, you should always make sure that the base of the rim and the interior of the tyre are free of foreign objects. Check the pressure after mounting or repairing a tyre. It is critical to your safety and to the wheelchair's performance that regulation air pressure be maintained and that tyres be in good condition.

## Possible Problems

### Wheelchair pulls to one side

- Check tyre pressure
- Check to make sure wheel turns easily (bearings, axle)
- Check to make sure both castors are making proper contact with the ground

### Castors begin to wobble

- Check to make sure all bolts are secure; tighten if necessary (see the page on torque)
- Check to make sure both castors are making proper contact with the ground

### Wheelchair / Cross-tube assembly does not snap into position in the seat saddle

- Chair is still new, i.e., the seat or backrest upholstery is still very stiff. This will improve with continued use.

### Wheelchair is difficult to fold up

- Screws of the cross-tube brackets are too tight.

### Wheelchair squeaks and rattles

- Check to make sure bolts are secure; tighten if necessary (see the page on torque)
- Apply small amount of lubrication to spots where movable parts come in contact with one another

### Wheelchair begins to wobble

- Check tyre pressure.
- Check to see if rear wheels are adjusted differently.

## Maintenance and Care

### Maintenance

- Check the tyre pressure every 4 weeks. Check all of the tyres for wear or damage.
- Check the brakes approximately every 4 weeks to make sure that they are working properly and easy to use.
- Change tyres as you would an ordinary bicycle tyre.
- All of the joints that are critical to using your wheelchair safely are self-locking nuts. Check every three months to make sure that all bolts are secure (See the page on torque). Self locking nuts should only be used once and should be replaced after single use.
- Use only mild household cleansers when your wheelchair is dirty. Use only soap and water when cleaning the seat upholstery.
- If your wheelchair should ever get wet, please dry it after use.
- A small amount of sewing-machine oil should be applied to quick-release axles approximately every 8 weeks. Depending on the frequency and type of use, we recommend taking your wheelchair to your authorised dealer every 6 months to have it inspected by trained personnel.

### CAUTION:

**Sand and sea water (or salt in the winter) can damage the bearings of the front and rear wheels. Clean the wheelchair thoroughly after exposure to these conditions.**

## Technical Data

### Total width:

- With standard wheels including handrims, narrow fitting: Seat width + 19,5 cm
- With drum brake wheels, handrims, narrow fitting: Seat width + 20 cm

### Folded dimensions:

- With standard wheels: approx. 33 cm
- Without standard wheels: approx. 17 cm

### Weight in kg:

- Wheelchair without footrest (24" rear wheels) from 16.5 kg
- Footrest (pair) 1.5 kg
- Standard sideguard 1.8 kg
- 24" rear wheel (solid) 1.4 kg

### Maximum load:

Approved to loads up to 125 kg (20 stone)

## Nameplates / Guarantee

### Rating plate

The rating plate is located on the cross-brace tube or cross-frame tube and also on the back of the user guide. This plate gives the exact type descriptions as well as other technical information. When ordering spare parts or making a complaint please give the following information

- Serial number
- Order number
- Month/Year

### Guarantee

You have purchased a high-quality Breezy product. As a sign of our gratitude, we are providing you with a 5-year guarantee on all frame components and the cross-tube assembly as well as 2 years on all other parts. We take no responsibility for any damage resulting from inappropriate or unprofessional assembly and/or repairs, through neglect and wear and tear, as well as any alterations to wheelchair components by the user or by any third party. In such cases, our product responsibility is voided.

**Custom wheelchairs cannot be exchanged.**

## Torque

**The torque for the screws shown:  
10 Nm castor and axle adapter screws must be secured with  
Loctite again after they have been loosened.**

**The torque for all screws is 7 Nm.**

Sunrise Medical GmbH + Co.KG  
Kahlbachring 2-4  
69254 Malsch/Heidelberg  
Deutschland  
Tel.: +49 (0) 72 53/980-0  
Fax: +49 (0) 72 53/980-111  
www.sunrisemedical.com

Sunrise Medical Limited  
High Street Wollaston  
West Midlands DY8 4PS  
England  
Phone: +44 (0) 13 84 44 66 88  
www.sunrisemedical.com

Sunrise Medical S.L.  
Polígono Bakiola, 41  
48498 Arrankudiaga – Vizcaya  
España  
Tel.: +34 (0) 9 02 14 24 34  
Fax: +34 (0) 9 46 48 15 75  
www.sunrisemedical.com

Sunrise Medical S.A.  
13, Rue de la Painguetterie  
37390 Chanceaux sur Choisille  
France  
Tel.: +33 (0) 2 47 55 44 00  
Fax: +33 (0) 2 47 55 44 03  
www.sunrisemedical.com

Sunrise Medical Srl  
Via Riva 20, Montale  
29100 Piacenza (PC)  
Italy  
Tel.: +39 05 23-57 31 11  
Fax: +39 05 23-5 70 60  
www.sunrisemedical.com

Sunrise Medical A.G.  
Lückhalde 14  
3074 Muri bei Bern  
Schweiz  
Tel.: +41 (0) 31-958-38 38  
Fax: +41 (0) 31-958-38 48  
www.sunrisemedical.com

Sunrise Medical AS  
Rehabsenteret  
1450 Nesoddtangen  
Norway  
Tel.: +47 (0) 66 96 38 00  
Faks: +47 (0) 66 96 38 38  
www.sunrisemedical.com

Sunrise Medical AB  
Britta Sahlgrens gata 8A  
421 31 Västra Frölunda  
Sweden  
Tel: +46 (0) 31-748 37 00  
Fax: +46 (0) 31-748 37 37  
www.sunrisemedical.com

Sunrise Medical B.V.  
Groningenhaven 18-20  
3433 PE Nieuwegein  
Nederlands  
Tel.: +31 (0) 30 60-8 21 00  
Fax: +31 (0) 30 60-5 58 80  
www.sunrisemedical.com

G2 A/S  
Graham Bells Vej 21-23 A  
8200 Aarhus N  
Denmark  
Tel.: +45 87 41 31 00  
Fax: +45 87 41 31 31  
www.G2.dk