ALLROUND 615

GEBRAUCHSANLEITUNG
DIRECTIONS FOR USE
NOTICE D'UTILISATION
GEBRUIKSHANDLEIDING
MANUALE PER L'USO
BRUGERVEJLEDNING

Vorwort

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Foreword (1.1.1)

Dear Customer,

We are very happy that you have decided in favour of a high-quality product from SUNRISE MEDICAL.

This user's manual will provide numerous tips and ideas so that your new wheelchair can become a trustworthy and reliable partner in your life.

Maintaining close links with our customers is of great importance to us at Sunrise Medical. We therefore like to keep you up-to-date with our new and current developments. Keeping **close** to our customers also means fast service when you need replacement parts or accessories, or just have a question about your wheelchair – and with as little red tape as possible.

We want you to be satisfied with our products and service. Sunrise Medical therefore constantly works at continuous development of its products. For this reason, changes can occur in our range of products with regard to shape, technology, and fittings. Consequently, no claims can be construed from the data or pictures contained in this user's manual.

SUNRISE MEDICAL has been awarded the ISO 9001 Certificate, which affirms the quality of our products at every stage, from R & D to production.

Please contact your local, authorised SUNRISE MEDICAL dealer if you have any questions concerning the use, maintenance, or safety of your wheelchair.

As the manufacturer, SUNRISE MEDICAL, declares that the lightweight wheelchairs conform to the 93/42/EEC / 2007/47/EEC guideline.

In the case that there is no authorised dealer in your area or you have any questions, you can contact Sunrise Medical either in writing or by telephone at:

Sunrise Medical Ltd.
Sunrise Business Park
High Street, Wollaston
West Midlands DY8 4PS
England
Telephone: +44/1384-446688
Fax: +44/1384-446699
www.sunrisemedical.co.uk

IMPORTANT:

DO NOT USE YOUR WHEELCHAIR UNTIL THIS MANUAL HAS BEEN READ AND UNDERSTOOD.

Table of Contents

Foreword

General safety notes and driving restrictions12-13

Wheelchair Components13

Handling

Quick-Release Axles14

Options

Tipping Tubes14

Brakes14

Extension for the Brake Lever14

Adjusting the Footrest14

Casters14

Seat Tilt Adjustment15

Seat Center of Gravity15

Rear Wheel15

Backrest Upholstery15

Adjustable Backrest Tilt15

Sideguards15

Push Handles16

Anti-Tip Tubes16

Adaptable Sling16

Crutch Holder16

Transit Wheels16

Tires and Mounting16

Possible Problems17

Technical Data17

Torque18

Use

Wheelchairs are exclusively for a user who is unable to walk or has limited mobility, for their own personal use in- and outdoor.

The maximum weight limit (includes the user and any weight of accessories fitted to the wheelchair) is marked on the serial number label, which is affixed to the crossbar or stabiliser bar below the seat.

Warranty can only be taken on if the product is used under the specified conditions and for the intended purposes.

The expected life of the wheelchair is 5 years.

Please **DO NOT** use or fit any 3rd party components to the wheelchair unless they are officially approved by Sunrise Medical.

Area of application

The variety of fitting variants as well as the modular design mean that it can be used by those who cannot walk or have limited mobility because of:

- Paralysis
- Loss of extremity (leg amputation)
- Extremity defect/deformity
- Joint contractures/joint injuries
- Illnesses such as heart and circulation deficiencies, disturbance of equilibrium or cachexia as well as for elderly people who still have strength in the upper body.

When considering provision, please also note the body size, weight, physical and psychological constitution, the age of the person, living conditions and environment.

General safety notes and driving restrictions (3.2)

The engineering and construction of this wheelchair has been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded in parts. Nevertheless, users may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must be strictly observed.

Unprofessional or erroneous changes or adjustments increase the risk of accident. As a wheelchair user, you are also part of the daily traffic on streets and pavements, just like anyone else. We would like to remind you that you are therefore also subject to any and all traffic laws.

Be careful during your first ride in this wheelchair. Get to know your wheelchair.

Before each use, the following should be checked:

- Quick-release axles on the rear wheels
- Velcro on seats and backrests
- Tyres, tyre pressure and wheel locks.

Before changing any of the adjustments of this wheelchair, it is important to read the corresponding section of the user's manual.

It is possible that potholes or uneven ground could cause this wheelchair to tip over, especially when riding uphill or downhill.

When riding over a step or up an incline frontally, the body should be leaning forward.

Danger!

NEVER exceed the maximum load of 125 kg for driver plus any items carried on the wheelchair. If you exceed the maximum load, this can lead to damage to the chair, or you may fall or tip over, lose control and may lead to serious injury of the user and other people.

Danger!

When it is dark, please wear light clothing or clothing with reflectors, so that you can be seen more easily. Make sure that the reflectors on the side and back of the wheelchair are clearly visible. We would also recommend that you fit an active light.

Danger!

To avoid falls and dangerous situations, you should first practice using your new wheelchair on level ground with good visibility.

Danger!

When getting on or off the wheelchair, do not use the footboards. These should be flipped up beforehand and swung to the outside as far as possible. Always position yourself as close as possible to the place where you wish to sit.

Danger!

Only use your wheelchair properly. For example, avoid travelling against an obstacle without braking (step, kerb edge) or dropping down gaps.

Danger!

The wheel locks are not intended to brake your wheelchair. They are only there to ensure that your wheelchair does not begin rolling unintentionally. When you stop on uneven ground, you should always use the wheel locks to prevent such rolling. Always apply both wheel locks; otherwise, your wheelchair could tip over.

Danger!

Explore the effects of changing the centre of gravity on the behaviour of the wheelchair, for example on inclines, slopes, all gradients or when overcoming obstacles. Do this with the secure aid of a helper.

Danger!

With extreme settings (e.g. rear wheels in the most forward position) and less than perfect posture, the wheelchair may tip over even on a level surface.

Danger!

Lean your upper body further forward when going up slopes and steps.

Danger!


Lean your upper body further back when going down slopes and steps. Never try to climb and descend a slope diagonally.

Danger!

Avoid using an escalator which may lead to serious injury in the event of a fall.

Danger!

Do not use the wheelchair on slopes $> 10^\circ$. The Dynamic safe slope is dependant on the chair configuration, the users abilities and the style of riding. As the users abilities and style of riding cannot be pre-determined then the max safe slope cannot be determined. Therefore this must be determined by the user with the assistance of an attendant to prevent tipping. It is strongly recommended that inexperienced users have Ant-tips fitted.

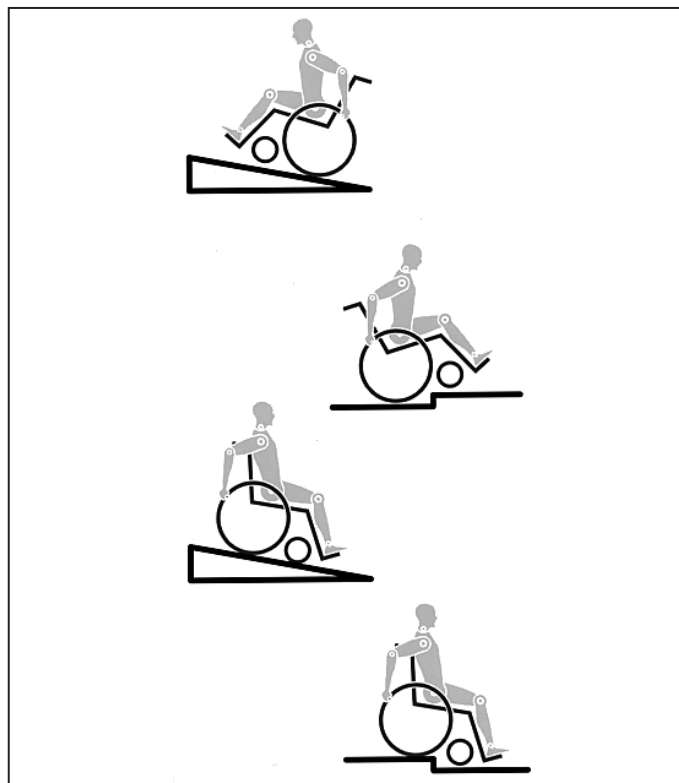Danger!

It is possible that potholes or uneven ground could cause this wheelchair to tip over, especially when riding uphill or downhill.

Danger!

Do not use your wheelchair on muddy or icy ground. Do not use your wheelchair where pedestrians are not allowed.

Danger!

To avoid hand injuries do not grab in between the spokes or between the rear wheel and wheel lock when driving the wheelchair.

Danger!

In particular when using lightweight metal handrims, fingers will easily become hot when braking from a high speed or on long inclines.

Danger!

Only attempt stairs with the help of an attendant. There is equipment available to help you, e.g. climbing ramps or lifts, please use them. If there is no such equipment available, then the wheelchair must be tipped and pushed, never carried, over the steps (2 helpers). We recommend that users over 100 kg in weight do not use this stairway manoeuvre!

In general, any anti-tip tubes fitted must be set beforehand, so that they cannot touch the steps, as otherwise this could lead to a serious tumble. Afterwards the anti-tip tubes must be set back to their correct position.

Danger!

Make sure that the attendant only holds the wheelchair using securely mounted parts (e.g. not on the footrests or the sideguards).

Danger!

When using the lifting ramp make sure that the anti-tip tubes fitted are positioned outside the danger area.

Danger!

Secure your wheelchair on uneven ground or when transferring (e.g. into a car) by using the brakes.

Danger!

If and whenever possible, during a journey in a specially fitted vehicle for disabled people, vehicle occupants should use the seats in the vehicle and the appropriate restraint system. This is the only way to ensure that occupants will have the maximum protection if there is an accident. When using safety elements offered by SUNRISE MEDICAL and using a specially designed safety system, lightweight wheelchairs can be used as a seat when being transported in a specially fitted vehicle. (See the Chapter on "Transportation").

Danger!

Depending on the diameter and setting of the castors, as well as

the centre of gravity setting of the wheelchair, the castors may begin to wobble at high speeds. This can lead to the castors being blocked and the wheelchair may tip over. Therefore, please make sure that the castors are adjusted correctly (see the Chapter "Castors"). In particular, do not travel on an incline without brakes, travel at a reduced speed. We recommend that novice users use anti-tip tubes.

Danger!

Anti-tip tubes should prevent the chair tipping over backwards unintentionally. Under no circumstances should they take the place of transit wheels, and be used to transport a person in a wheelchair with the rear wheels removed.

Danger!

When reaching for objects (which are in front of, to the side or behind the wheelchair) make sure that you do not lean too far out of the wheelchair, as if you change the centre of gravity there is a risk of tipping or rolling over. The hanging of additional load (back pack or similar items) onto your chair backposts can affect the rearward stability of your chair, especially when used in combination with recliner backrests. This can cause the chair to tip backwards causing injury.

Danger!

For thigh amputees you must use anti-tip tubes.

Danger!

Before setting off, check that your tyre pressure is correct. For rear wheels it should be at least 3.5 bar (350 kPa). The max. pressure is indicated on the tyre. The knee-lever brakes will only work if there is sufficient tyre pressure and if the correct setting has been made (see the Chapter on "Brakes").

Danger!

If the seat and back sling are damaged, you must replace them immediately.

Danger!

Be careful with fire, in particular with burning cigarettes. Seat and back slings can be set alight.

Danger!

If the wheelchair is subject to direct sunlight for a long period of time, then parts of the wheelchair (e.g. frame, legrests, brakes and sideguard) may become hot (>41°C).

Danger!

Always make sure that the quick-release axles on the rear wheels are set properly and lock in. If the button on the quick-release axle is not pressed in, the rear wheel cannot be removed.

Note !

The effect of the knee-lever brake as well as the general driving characteristics are dependant on tyre pressure. The wheelchair is significantly lighter and easier to manoeuvre when the rear wheels are pumped up correctly and both wheels have the same pressure.

Note !

Make sure that your tyres have sufficient tread! Please note that you are subject to any and all traffic laws when driving in public traffic.

Note !

Always be careful with your fingers when working or adjusting the wheelchair!

The products shown and described in this manual may not be exactly the same in every detail as your own model. However, all instructions are still entirely relevant, irrespective of detail differences. The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

Transportation

Your wheelchair is not suitable for use as a seat in vehicles.

Do not sit in the wheelchair during transportation. Use the vehicle seat or a properly secured transport approved seat.

Wheelchair Components(4.1)

We at SUNRISE MEDICAL have been awarded the ISO 9001 Certificate, which affirms the quality of our products at every stage, from R & D to production. This product complies with the standards set forth in EU directives. Optional equipment and accessories are available at extra charge.

Wheelchairs:

1. Push handles
2. Backrest sling
3. Sideguards
4. Seat sling
5. Footrest
6. Casters
7. Caster plates
8. Fork
9. Quick-release axles
10. Wheel locks
11. Handrim
12. Rear wheel

Handling

Quick-Release Axles for Rear Wheels (6.2)

The rear wheels are equipped with quick-release axles. The wheels can thus be installed or removed without using tools. To remove a wheel, simply depress the quick-release button on the axle (1) and pull it out.


04

CAUTION:

Hold the quick-release button on the axle depressed when inserting the axle into the frame to mount the rear wheels. Release the button to lock the wheel in place. The quick-release button should snap back to its original position.

CAUTION:

After each adjustment of the rear wheels, check the interval to the wheel locks and readjust if necessary.

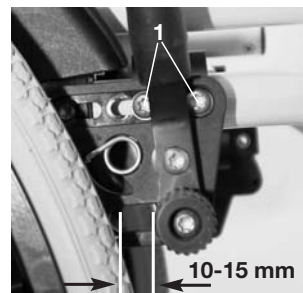10.4 Standard brake

Extension for the Brake Lever (7.11)

The extension for the brake lever can be removed or folded down. The longer lever helps to minimize the effort needed to set the wheel locks.

Caution:

Mounting the wheel lock too close toward the wheel will result in a higher effort to operate. This might cause the brake lever extension to break!

11

Leaning onto the brake lever extension while transferring will cause the lever to break! Splashing water from tires might cause the wheel lock to malfunction.

Options – Tipping Tubes

Tipping Tubes (7.1)

Tipping tubes are used by attendants to tip a wheelchair over an obstacle. Simply step on the tube to push a wheelchair, for example, over a curb or step.

07

Options – Wheel Locks

Wheel Locks (7.10)

Your wheelchair is equipped with two wheel locks. They are applied directly against the tires. To engage, press both wheel-lock levers forward against the stops (Fig. 10.1 and 10.3). To release the wheel, pull the levers back to their original positions.

Braking power will decrease with:

- Worn tire tread
- Tire pressure that is too low
- Wet tires
- Improperly adjusted wheel locks.

The wheel locks have not been designed to be used as brakes for a moving wheelchair. The wheel locks should therefore never be used to brake a moving wheelchair. Always use the hand-rims for braking. Make sure that the interval between the tires and wheel locks complies with given specifications (Fig. 10.2 and 10.4). To readjust, loosen screw (1) and set the appropriate interval. Tighten screw (see the page on torque)

10.1 Knee-lever brake

10.2 Knee-lever brake

10.3 Standard brake

Options – Adjusting the Footrest

Adjusting the Footrest (7.25)

Removing screw (1) will allow you to adjust the footrest to correspond to the length of your lower leg. The angle of the footrest may be individually adjusted (not rigid footrest) by loosening nuts (2). The footrest bracket (3) prevents feet from unintentionally slipping out place. When finished, make sure that all screws have been properly tightened (see the page on torque).


18

Options – Casters

Castors (7.43)

Because seat adjustments are made directly on the seat frame, you will not need to adjust the caster mounts. However, to maintain the fork at a constant angle of 90°, the casters can only be mounted in the holes provided (see picture). When finished, make sure that all screws have been properly tightened (see the page on torque).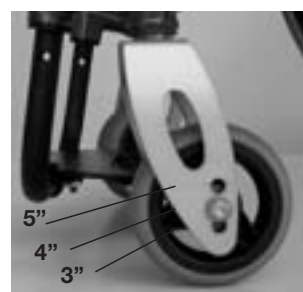


24

Options – Seat Tilt Adjustment

Seat Tilt Adjustment (7.53)

The sideguard must be removed before adjusting the seat tilt in order to reach the necessary screws. Once the collarnuts (1) and screws (2) on the rear seat mount have been removed, the angle at which the seat is tilted can be adjusted. When the seat is tilted at the desired angle, screw on the collars from the outside and screws from the inside.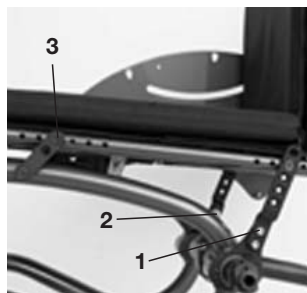

30


Adjusting Seat Height

Adjust the rear seat height as described above. The front seat mount can be adjusted to the desired height after removing the screws (3).

Options – Seat Center of Gravity

Seat Center of Gravity (7.54)

The position of the body's center of gravity with respect to the rear wheels is critical for ensuring easy and pleasant use of the wheelchair. Several positions are possible: moving the seat frame (1) forward along the holes drilled in the frame (2 through 6) will make the chair stable, but it will not be as maneuverable as it would be if the seat frame were positioned farther back. When finished, make sure that all screws have been properly tightened (see the page on torque).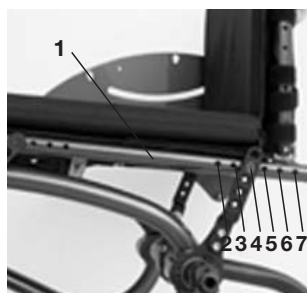

31


CAUTION:

The wheelchair is liable to tip when the quantity used to measure center of gravity (see picture) is at its greatest. For reasons of safety, we therefore recommend that an anti-tip tube be mounted in the drill hole (6) when assembling the frame; this will prevent the wheelchair from tipping backwards when going over larger obstacles or curbs.

Options – Rear Wheels

Rear Wheels (7.60)

The distance between the rear wheels and the sideguard can be adjusted as follows: Loosen the axle sleeve (1) by loosening the screws (2). Pull the axle sleeve (1) out to the desired distance (no more than 3 cm). To ensure that the wheel alignment has been properly adjusted, check to make sure that the pin can be seen in the groove. When finished, make sure that all screws have been properly tightened (see the page on torque).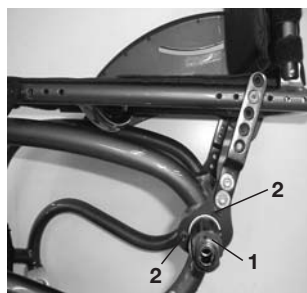

30.1

Options – Backrest Upholstery

Adjustable Backrest Upholstery (7.72)

The sling can be adapted to any degree of tension by adjusting the Velcro fasteners. The padding in the backrest can be removed through an interior opening, allowing you to cushion the backrest according to your individual needs.

Options – Adjustable Backrest Tilt

Adjustable Backrest Tilt (7.73)

Backrest Tilt

First change the rear height of the sideguard by removing the screws (1). Regardless of the seat frame position (seat height), the sideguard must be adjusted to that position in order to prevent the clothing guard from rubbing against the wheel. Next adjust the backrest (2) to the desired angle. Several holes (4) have been drilled into the front portion of the sideguard (3) to set the backrest at the desired angle.

Backrest Height

After you have removed the rear screws (1), you can remove the sideguard. By pulling up on the backrest frame, you can adjust the height by a total of 5 cm in 2.5 cm increments. When finished, make sure that all screws have been properly tightened (see the page on torque).

CAUTION:

The screws (1) connect the backrest directly to the frame; screwing them in too tightly may cause them to break. Conversely, screws that are too loose can be easily lost. In either case, injuries could result when ascending stairs.

For your own safety, we recommend that adjustments only be made by authorized dealers.

Options – Sideguards

Sideguards (7.84)

The sideguards can be unlocked by pulling up on the rear cross bar (1). Swinging the backrest (2) forward will allow it to fold down onto the seat.


CAUTION:

Neither the sideguards nor the armrests are to be used for lifting or carrying the wheelchair.

Options – Push Handles

Height-Adjustable Push Handles (7.96)


These handles are secured with pins to prevent them from sliding out unintentionally. Opening the quick-release lever (1) makes it possible to adjust the push handles to meet your individual needs. As you move the lever, you will hear a locking mechanism; you may now easily position the push handle as desired. The nut (2) on the tension lever determines how tightly the push handles are clamped into place. If the nut is loose after adjusting the tension lever, the push handle will also be too loose. Turn the push handle from side to side before use to make sure that it is clamped securely enough into place. After adjusting handle height, always clamp the tension lever (1) securely into place. If the lever is not secure, injuries could result when lifting over obstacles.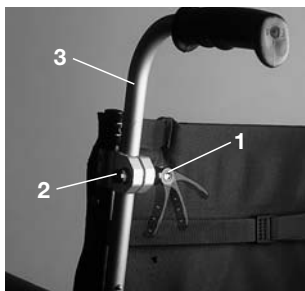

49

Fold-Down Push Handles (7.91)

If the push handles are not in use, they can be folded down by depressing the button (2). When they are needed again, simply flip them back up until they click into place.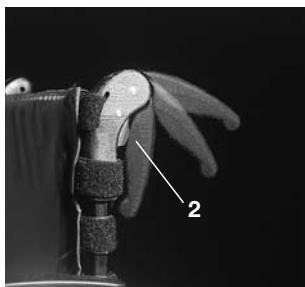


50

Options – Anti-Tip Tubes

Anti-Tip Tubes (7.93)

Anti-tip tubes (1) provide additional safety for inexperienced users when they are still learning how to operate their wheelchairs. Anti-tip tubes (1) prevent a wheelchair from tipping over backwards. Pushing the anti-tip tubes (1) will move them downwards; they can also be swung forward. An interval of 3 cm to 5 cm should be maintained between them and the ground. You must swing the anti-tip tubes forward when going up and down large obstacles (such as a curb) to prevent them from touching the ground.

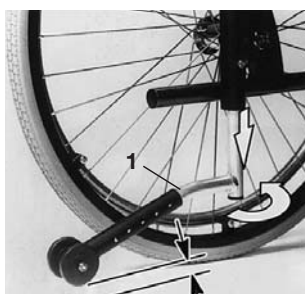52

53

Options – Adaptable Sling

Adaptable Sling (7.100)

The sling can be adapted to any degree of tension by adjusting the Velcro fasteners. These fasteners (2) can be exposed by removing the upper seat sling (1). Loosen the fasteners at the buckles (3) to adjust the sling directly to any position. After the fasteners are tightly reapplied, they will prevent the sling from slipping.

Options – Crutch Holder

Crutch Holder (7.110)

This device permits crutches to be transported directly on a wheelchair. It has a Velcro loop (1) to fasten crutches or other aids.

CAUTION:

Never try to use or even remove the crutches or other aids while moving.

Options – Transit Wheels

Transit Wheels (7.113)

Transit wheels should be used wherever your wheelchair would be too wide if the rear wheels were used (e.g., in airplanes, buses, etc.). After the rear wheels have been removed with the aid of the quick-release axles, the transit wheels can immediately be used to continue riding. The transit wheels are mounted so that they are approx. 3 centimeters above the ground when not in use. They are thus out of the way when riding, transporting, or when tipping to pass over obstacles (e.g., curbs, steps, etc.).

CAUTION:

Your wheelchair does not have any wheel locks when the transit wheels are being used.

49.1

49.2

Tires and Mounting

Tires and Mounting (8.1)

Always make sure you that you maintain the correct tire pressure, as this can have an effect on wheelchair performance. If the tire pressure is too low, rolling resistance will increase, requiring more effort to move the chair forward; low tire pressure also has a negative impact on maneuverability. If the tire pressure is too high, the tire could burst. The correct pressure for a given tire is printed on the surface of the tire itself. Tires can be mounted the same way as an ordinary bicycle tire. Before installing a new inner tube, you should always make sure that the base of the rim and the interior of the tire are free of foreign objects. Check the pressure after mounting or repairing a tire. It is critical to your safety and to the wheelchair's performance that regulation air pressure be maintained and that tires be in good condition.

Possible Problems

Wheelchair pulls to one side (9.1)

- Check tire pressure
- Check to make sure wheel turns easily (bearings, axle)
- Check angles of casters
- Check to make sure both casters are making proper contact with the ground

Casters begin to wobble (9.2)

- Check angle of casters
- Check to make sure all bolts are secure; tighten if necessary (See the page on torque)
- Check to make sure both casters are making proper contact with the ground

Wheelchair squeaks and rattles (9.6)

- Check to make sure bolts are secure; tighten if necessary (see the page on torque).
- Apply small amount of lubrication to spots where movable parts come in contact with one another

Wheelchair begins to wobble (9.7)

- Check angle at which casters are set
- Check tire pressure
- Check to see if rear wheels are adjusted differently

Maintenance and Care

Maintenance (9.9)

- Check the tire pressure every 4 weeks. Check all of the tires for wear or damage.
- Check the brakes approximately every 4 weeks to make sure that they are working properly and easy to use.
- Change tires as you would an ordinary bicycle tire.
- All of the joints that are critical to safe using your wheelchair safely are self-locking nuts. Please check every three months to make sure that all bolts are secure (See the page on torque). Safety nuts should only be used once and should be replaced after multiple uses.
- Please use only mild household cleansers when your wheelchair is dirty. Use only soap and water when cleaning the seat upholstery.
- If your wheelchair should ever get wet, please dry it after use.
- A small amount of sewing-machine oil should be applied to quick-release axles approximately every 8 weeks. Depending on the frequency and type of use, we recommend taking your wheelchair to your authorized dealer every 6 months to have it inspected by trained personnel.

CAUTION:

Sand and sea water (or salt in the winter) can damage the bearings of the front and rear wheels. Clean the wheelchair thoroughly after exposure.

Technical data

Seatheight: (10.615)

The available seatheights depend on what kind of frameheight (high, low), what kind of rear seatheight (low, medium, high) and rearwheelsize (24" or 26") is chosen

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* When using a 26" rearwheel these measurements are 2 cm higher.

Lower leg

Depending on what kind of frame (low, high) and footrest, seat-width, are possible the following length: The chart is relevant for a frametype „low“. When choosing frame „high“ this measure increases the possible leglength adjustments by 4 cm!

Total width:	24"	26"
with 0° camber:	SB+ 14 cm	SB+ 14 cm
with 4° camber:	SB+ 21 cm	SB+ 24 cm
with 7° camber:	SB+ 27 cm	SB+ 29 cm
with 10° camber:	SB+ 32 cm	SB+ 34 cm
with 14° camber:	SB+ 39 cm	SB+ 41 cm

Maximum weight limit:

Approved for user weights up to 140 kg

Nameplates

Nameplates (11.1)

The nameplate is located on either the cross-tube assembly or the transverse frame tube, as well as on the back page of the owner's manual. The nameplate indicates the exact model designation and other technical specifications. Please provide the following pieces of information whenever you have to order replacement parts or to file a claim:

- Serial number
- Order number
- Month / Year

Guarantee

You have purchased a high-quality Sopur product. As a sign of our gratitude, we are providing you with a 5-year guarantee on all frame components and for the cross-tube assembly. We are not responsible for any damage resulting from inappropriate or unprofessional installation and/or repairs, neglect, wear, or from changes in any wheelchair components caused either by the user or by third parties. In such cases, this guarantee shall be considered null and void.


Custom wheelchairs cannot be exchanged.

Caster size	Platform Footrest			Footrest with bumper		
	3"	4"	5"	3"	4"	5"
Seatwidth 32 cm	36-43 ¹ 23-46 ²	N/A ¹ 23-36 ²	N/A ¹ 23-34 ²	24-39	24-39	24-39
Seatwidth 34 cm	36-46 ¹ 23-46 ²	36-38 ¹ 23-36 ²	N/A ¹ 23-32 ²	24-39	24-39	24-39
Seatwidth 36 cm	36-46 ¹ 23-46 ²	36-38 ¹ 23-36 ²	36-38 ¹ 23-34 ²	24-39	24-39	24-39
Seatwidth 38 cm	36-46 ¹ 23-46 ²	36-46 ¹ 23-40 ²	36-38 ¹ 23-36 ²	24-39	24-39	24-39
Seatwidth 40 cm	36-46 ¹ 23-46 ²	36-46 ¹ 23-40 ²	36-38 ¹ 23-38 ²	24-39	24-39	24-39
Seatwidth 42 cm	36-46 ¹ 23-46 ²	36-46 ¹ 23-40 ²	36-40 ¹ 23-38 ²	24-39	24-39	24-39
Seatwidth 44 cm	36-46 ¹ 23-46 ²	36-46 ¹ 23-40 ²	36-46 ¹ 23-38 ²	24-39	24-39	24-39


Caster size	Footrest with bar			Footrest, fold-up		
	3"	4"	5"	3"	4"	5"
Seatwidth 32 cm	36-46	36-46	36-46	–	–	–
Seatwidth 34 cm	36-46	36-46	36-46	–	–	–
Seatwidth 36 cm	36-46	36-46	36-46	–	–	–
Seatwidth 38 cm	36-46	36-46	36-46	40-48	40-48	–
Seatwidth 40 cm	36-46	36-46	36-46	40-48	40-48	–
Seatwidth 42 cm	36-46	36-46	36-46	40-48	40-48	–
Seatwidth 44 cm	36-46	36-46	36-46	40-48	40-48	–

(1) Platform Footrest „low mount“, (2) Platform Footrest „high mount“


Torque (12.1)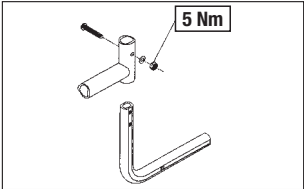

67

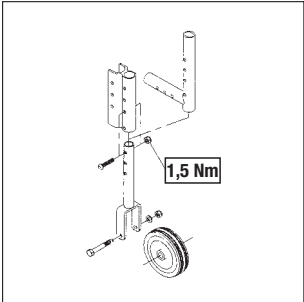70

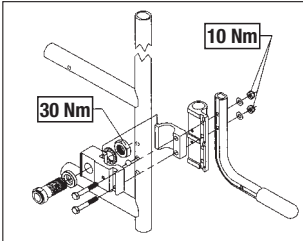73

The torque for the M6 screw is 7 Nm, unless otherwise specified.

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Sunrise Medical AB
Sunrise Medical B.V.
G2 A/S