




	Empulse® R90® Actuator Replacement Instruction Sheet 2
	Výměna pohonu Empulse® R90® Návod k použití 4
	Empulse® R90® Austausch des Stellmotors Anleitung 6
	Instruktionsark til Empulse® R90® aktuatorudskiftning..... 8
	Hoja de instrucciones correspondiente al sustitución del actuador del Empulse® R90® 10
	Fiche d'instruction - Remplacement du vérin Empulse® R90® 12
	Istruzioni per la sostituzione dell'attuatore di Empulse® R90® 14
	Vervanging Actuator Empulse® R90® Instructies 16
	Instruksjonsark for Utskiftning av Empulse® R90®-aktuator 18
	Instrukcje użytkowania Wymiana siłownika Empulse® R90® 20
	Folha de instruções do substituição do atuador Empulse® R90® 22
	Instruktionsblad för Empulse® R90® byte av ställdon 24
	Návod na použitie k pohonu Empulse® R90® 26

Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support.

Tools required

- | | |
|--|--|
| 1. Allen wrench, sizes: 2.5 mm,
3mm | 3. Torque wrench |
| 2. Pliers | 4. 8mm socket
(6-point or 12-point) |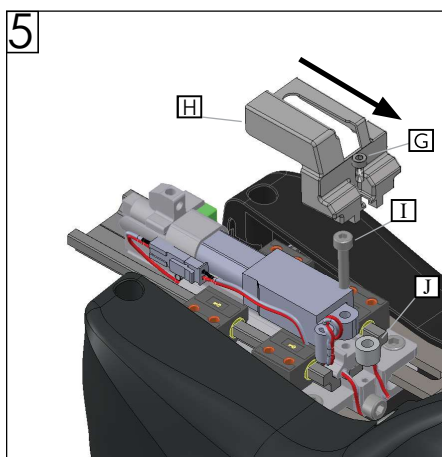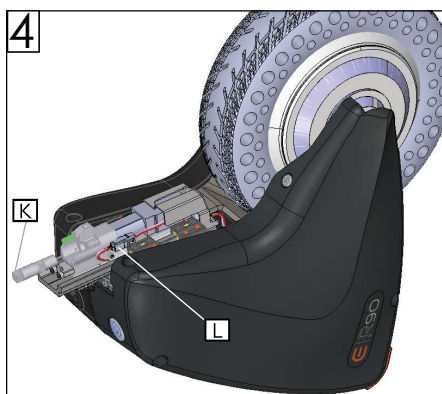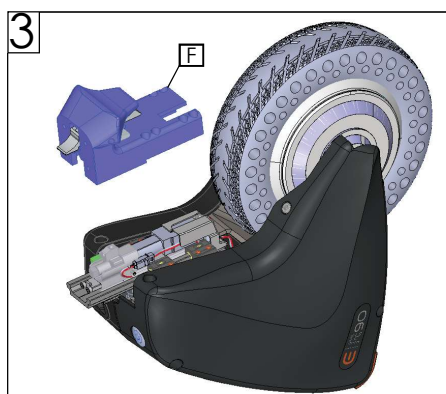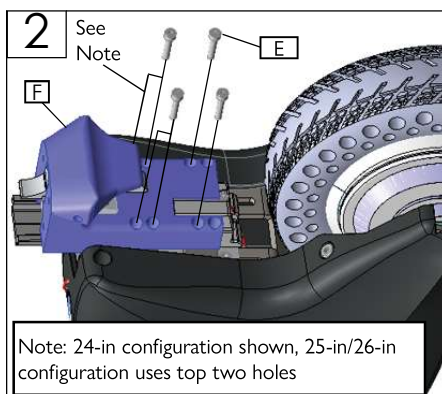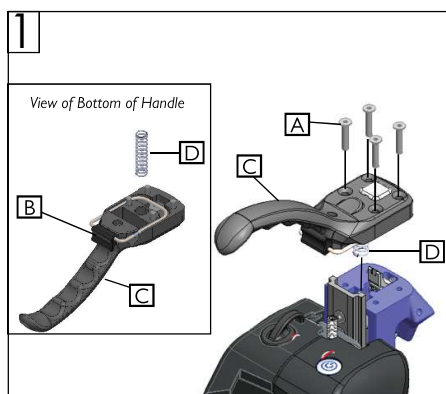

1. To remove handle: (Fig. 1)
 - a. Hold the handle in place so the spring does not pop up once the screws are removed.
 - b. Using a 3mm Allen wrench, remove the four screws (A) that hold the handle (C) and the latch wire holder (B).
 - c. Remove handle (C) and spring (D).

To complete steps 2-5, set the device in a horizontal position (leave battery in, battery not shown).

2. Using a 3mm Allen wrench, remove the front bolts (E) that mount the latch block (F). Note the position of the bolts for reassembly (Fig. 2)
3. Remove the latch block (F). (Fig. 3)
Note: There is a spring in the latch block. It must be positioned correctly for reassembly.
4. Next, remove the actuator.
 - a. Back off the actuator bolt (K) by hand or with a plier. (Fig. 4)
 - b. Disconnect the connector (L) of the actuator harness. Then, remove the actuator. (Fig. 4)
5. Next, remove the cover and actuator hardware.
 - a. Remove the cover bolt (G) and cover (H) by sliding toward the wheel using a 2.5mm Allen wrench. (Fig. 5)
 - b. Remove the bottom actuator bolt (I) using a 3mm Allen wrench. Next, remove the spacer (J) underneath the actuator. (Fig. 5)








DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Sunrise Medical Technical Service Department before operating and/or servicing the Empulse device. Failure to do so may result in damage and/or injury. **Find more information and important warnings in the device owner's manual or at:**

www.SunriseMedical.co.uk

R90® ACTUATOR REPLACEMENT INSTRUCTION SHEET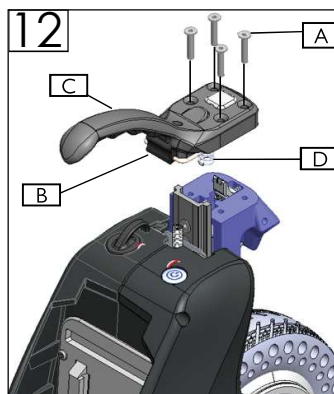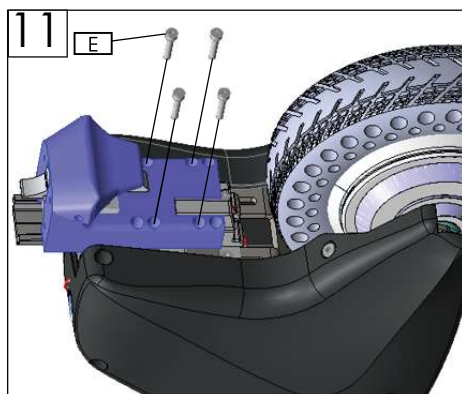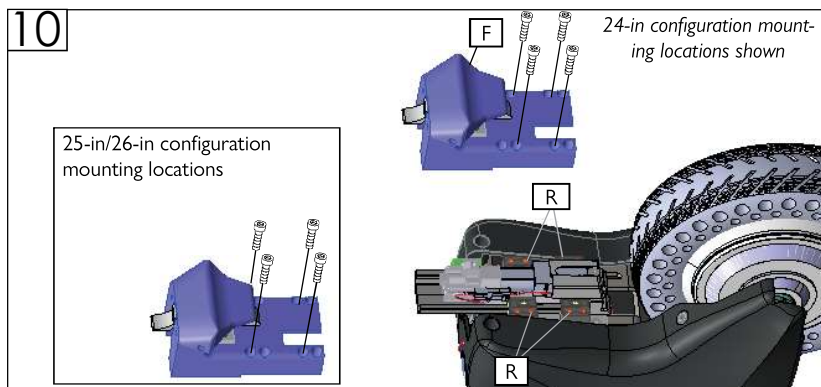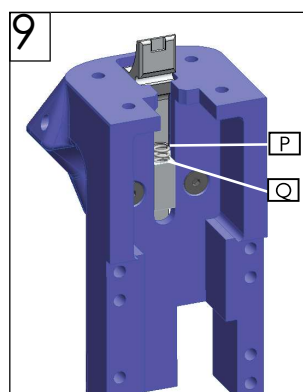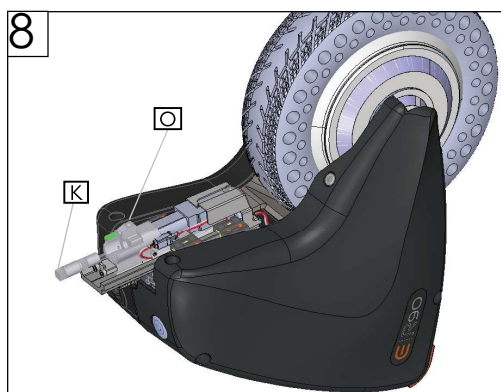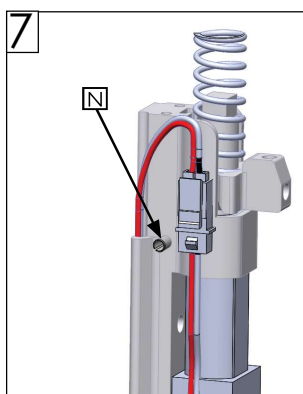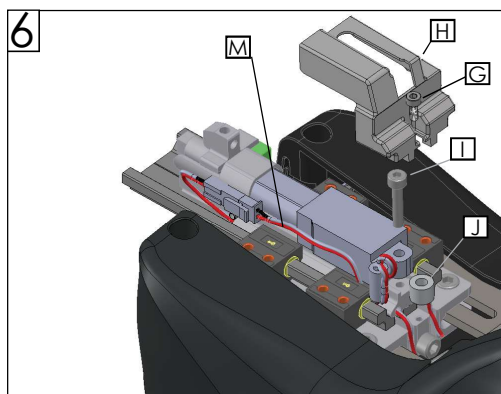

To mount the new actuator, follow steps 1-5 in reverse order.

6. Insert the actuator as shown and the wire should be on the side as shown. (Fig. 6). Insert the spacer (J) back under the actuator before inserting bolt (I). Using a 3mm Allen wrench, fasten the M4 bolt (I) and torque it to 2.2 Nm. Then, slide the cover (H) back into the rail over the actuator, and ensure the ferrite is under the cover toward the back of the actuator and torque M3 bolt (G) to 1.0 Nm using a 2.5mm Allen wrench. Ensure the wire (M) is not pinched (Fig. 6)
7. Ensure the connector is inline with the roll pin (N) (Fig. 7)
8. Thread the actuator bolt (K) into the actuator through the insert (O) and torque to 2.8 Nm. To help ease the assembly of the latch block, activate the actuator to push out the insert to around 5mm by pressing the up and down button and immediately pulling the battery out to stop the actuator. (Fig. 8)
9. Ensure the spring (P) is still positioned in the pocket in the latch block (Q). (Fig. 9)
10. Reinstall the battery and lay the device down horizontal to reassemble the latch block. Align the slider blocks (R) with latch block (F). (Fig. 10) Tug the connector inside the latch block cutout at the same time. To ease the installation of the slider blocks, use the 2.5mm Allen tool to align them using the second holes.
11. Fasten the M4 hardware (E) using a 3mm Allen wrench and torque to .7 Nm. (Fig. 11)
Note: Refer to notes from step 2 for correct mounting position.
12. Insert the battery all the way back, and push the up and down button two times and the actuator will extend and retract all the way home. Place the spring (D) into the actuator bolt. Then, hold the latch wire holder (B) on the handle (C) and insert the handle into the latch block. Please note that it requires significant strength because of the spring; as an alternative option use a zip tie to hold the handle in place. Torque the screws (A) to 2.8 Nm. (Fig. 12)

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S