

E | F35

STEP-BY-STEP DOCKING

- Always make sure that the device is switched off before docking and undocking
- Always apply the wheelchair brakes
- Ensure the wheelchair is fully connected with the pull device before use
- As with all moving parts, there is a risk that fingers and / or clothing could become trapped
- Be extra careful when operating the docking clamps



1

Insert the subframe into the docking clamps.



2

Now rotate the tommy screws of the clamps in a clockwise direction until they are hand tight and the intermediate sub-frame is securely held in the clamps.



3

Take the pull device and insert the docking pin ends into the central bracket of the sub-frame.

4

Push the handlebar forwards/ upwards until the locking bolt automatically clicks into place. Only release the wheelchair brakes once device is locked in place.



Empulse




Scan the QR code to see the Service Videos!

E | F35

STEP-BY-STEP UNDOCKING

- Always make sure that the device is switched off before docking and undocking
- Always apply the wheelchair brakes
- Ensure the wheelchair is fully connected with the pull device before use
- As with all moving parts, there is a risk that fingers and / or clothing could become trapped
- Be extra careful when operating the docking clamps

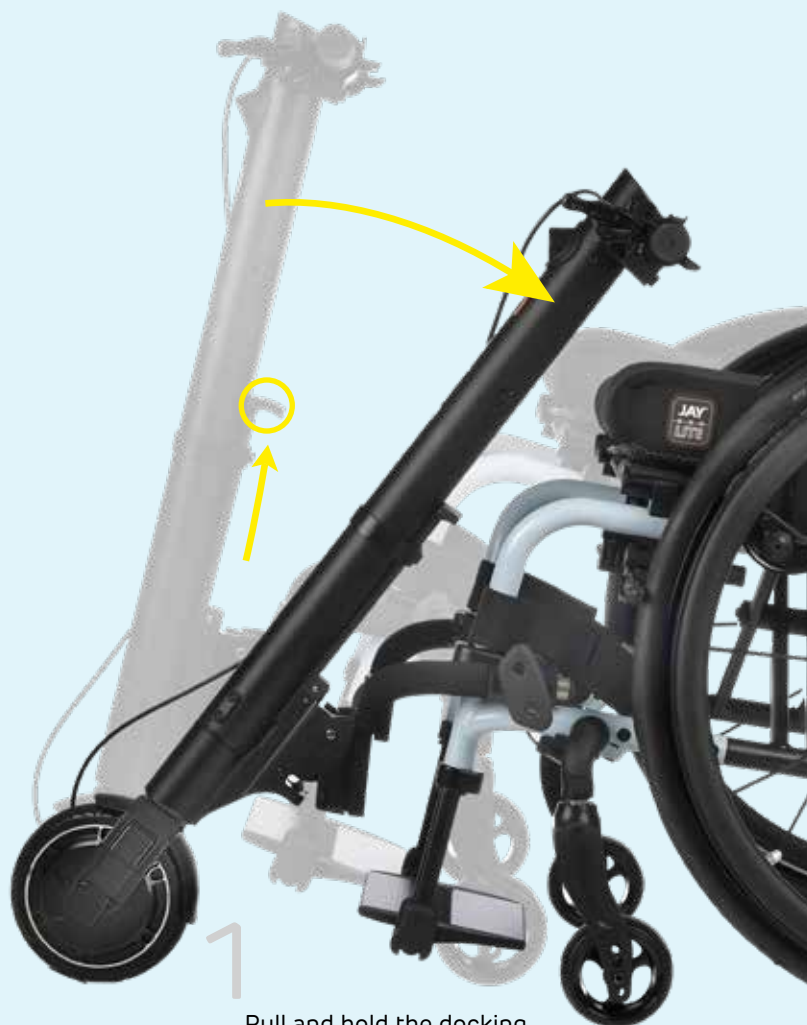

1
Pull and hold the docking release lever on the main tube. Now pull the handlebar gently backwards/downwards.



2
Now lower the pull device slowly until the castors of the wheelchair touch the floor. To remove the F35 from the subframe only, steps 1 and 2 can be performed independently of steps 3 and 4.



3
Undo the tommy screws of the clamps on both sides and pull the subframe until the tubes of the intermediate sub-frame have been pulled completely out of the clamps.



4
Take out the subframe from the docking clamps.



Scan the QR code to see the Service Videos!