

Empulse®

E | R10

ATTENDANT CONTROL MOUNT INSTRUCTIONS FOR CHAIRS WITH DRUM BRAKES



BEFORE



AFTER



When fitting the Empulse R10 to a Breezy BasiX with Drum brakes, please modify the mounting instructions as follows to ensure ease of use. When the mount is installed as per instructions, the drum brake levers block fitment as shown in the left image.

Instead unscrew the mount, rotate the mount 180° and re fasten to the handle as shown in the image on the right. This will avoid the receiver being blocked by the drum brake levers