

How to fit your stick clip



Part code:
HX15151R

Keep your reacher/grabber with you wherever you go.



Intended use: This stick clip is specifically designed to fit snugly on all 'Classic' reachers.

To fit onto your reacher



Pinch the 'rabbit ears' to open the clip



Push onto the top of the reacher bar



Make sure it 'clunk-clicks' into place

Clip it onto a crutch, chair leg
walking pole/stick/frame or your
wheelchair - simple.





The Helping Hand Company Ltd
Bromyard Road, Ledbury, HR8 1NS United Kingdom
+44 (0)1531 635678
sales@HelpingHand.co.uk



+44 (0)1531 635678
sales@helpinghand.co.uk
helpinghand.co.uk

