



Breezy 300

Spare Parts Catalogue

 **Breezy**

**INNOVATE.
BUILD FOR THE FUTURE**



This product is manufactured to comply with the Medical Device Directive 93/42/EEC.

Use of Illustrations & Text:

The illustrations and accompanying text set out in this document are the property of Sunrise Medical Limited and must not be copied or used for manufacturing, tendering or any other purpose without written consent from Sunrise Medical.

Ordering:

Item numbers shown as N/A are for reference purposes only and should not be used when ordering replacement parts. Always quote the correct serial number and part number.

Quality Management Systems:

Sunrise Medical Limited is accredited to ISO 9001: 2000, which is the internationally recognised standard for quality management systems, as well as ISO 13485: 2003, which specially governs Medical Devices. These approvals ensure that we provide quality in all areas of our business from development through to final delivery.

Parts Disclaimer:

It is potentially hazardous to fit or use any parts other than genuine Sunrise Medical parts. The company disclaims all liability for the consequences of such use, which in addition voids the product warranty.

This product is for the carriage of one person and must not under any circumstances be used for any other purpose.

Queries:

Should you have any queries then please contact Sunrise Medical Ltd on

Tel: +44 (0)1384 44 66 66


Fax: +44 (0)1384 44 66 11

Breezy 300	5
Frame.....	5
Breezy 300 - Cross-brace	5
Breezy 300 - Side frame, up until 07/05/2012 (no. 30012050005)	6
Breezy 300 - Side frame from 07/05/2012 (no. 30012050005).....	8
Breezy 300 - Backrest tubes up until 07/05/2012 (no. 30012050005)	9
Breezy 300 - Backrest tubes from 07/05/2012 (no. 30012050005)	11
Breezy 300 - Back tube - Reclining model	12
Breezy 300 - Axle plate for wheels and backrests	13
Seat.....	14
Breezy 300 - Seat upholstery.....	14
Breezy 300 - Anatomical seat	15
Breezy 300 - Anatomical seat with toilet bucket.....	16
Backrests	17
Breezy 300 - Backrest Upholstery.....	17
Breezy 300 - Two-Part Backrest upholstery.....	19
Breezy 300 - Reclining Backrest Upholstery.....	20
Breezy 300 - Anatomical backrest	22
Breezy 300 - Stiffener tube	23
Breezy 300 - Height-adjustable push handles	24
Armrests.....	25
Breezy 300 - Armrests	25
Breezy 300 - Armrests for reclining backrest	28
Breezy 300 - Height adjustable armrests	30
Hangers	31
Breezy 300 - Footrests up until 07/05/2012 (no. 30012050005).....	31
Breezy 300 - Footrests from 07/05/2012 (no. 30012050005)	33
Breezy 300 - Short footrests	34
Breezy 300 - Elevating legrests up until 07/05/2012 (no. 30012050005)	35
Breezy 300 - Elevating legrests from 07/05/2012 (no. 30012050005).....	37
Breezy 300 - Fixed footrests	39
Breezy 300 - Footboard up until 07/05/2012 (no. 30012050005)	40
Breezy 300 - Footboard from 07/05/2012 (no. 30012050005).....	42
Breezy 300 - Continuous footboard up until 07/05/2012 (no. 30012050005)	43
Breezy 300 - Continuous footplate from 07/05/2012 (no. 30012050005)	44
Breezy 300 - Angle-adjustable footplate	45
Castors.....	46
Breezy 300 - 6" solid wheel.....	46
Breezy 300 - 8" x 2" pneumatic tyre	47
Breezy 300 - 8" x 1" solid wheel.....	48
Breezy 300 - 8" x 2" solid wheel.....	49
Breezy 300 - Fork	50
Rear Wheels	51
Breezy 300 - Sport 24" solid wheel	51
Breezy 300 - Sport 24" pneumatic tyre	52
Breezy 300 - 24" solid spoke wheel	53
Breezy 300 - 24" pneumatic wheel with spokes.....	54

Table of Contents


Breezy 300 - 22" pneumatic wheel with spokes.....	55
Breezy 300 - Quick Release Axle	56
Breezy 300 - 12" solid wheel.....	57
Breezy 300 - 12" pneumatic tyre	58
Breezy 300 - Hemiplegic Wheel.....	59
Wheel Locks	60
Breezy 300 - Brakes up until 07/05/2012 (no. 30012050005).....	60
Breezy 300 - Brakes from 07/05/2012 (no. 30012050005)	61
Breezy 300 - Brake Extension	63
Breezy 300 - Drum brakes for 12" wheel up until 07/05/2012 (no. 30012050005)	64
Breezy 300 - Drum brakes for 12" wheel from 07/05/2012 (no. 30012050005).....	66
Breezy 300 - Drum brakes for 24" wheel	67
Headrest	68
Breezy 300 - Standard headrest	68
Breezy 300 - Standard headrest for anatomic backrest.....	69
Breezy 300 - Standard headrests for reclining chairs	70
Breezy 300 - Anatomic headrest.....	71
Breezy 300 - Anatomic headrest for anatomic backrest	72
Breezy 300 - Anatomic headrests for reclining chairs.....	73
Breezy 300 - Back Extension (for reclining backrest)	74
Accessories.....	75
Breezy 300 - Eating and Reading Table	75
Breezy 300 - Drip support	76
Breezy 300 - Basic drip support.....	77
Breezy 300 - IV Support with multiposition clamp.....	78
Breezy 300 - Crutch Holder.....	79
Breezy 300 - Tank holder.....	80
Breezy 300 - Double Amputee Support	81
Breezy 300 - Transit wheel	82
Breezy 300 - Anti-Tip Tubes	83
Breezy 300 - Unfolding device	84
Breezy 300 - Transportation kit.....	85

Breezy 300 - Cross-brace

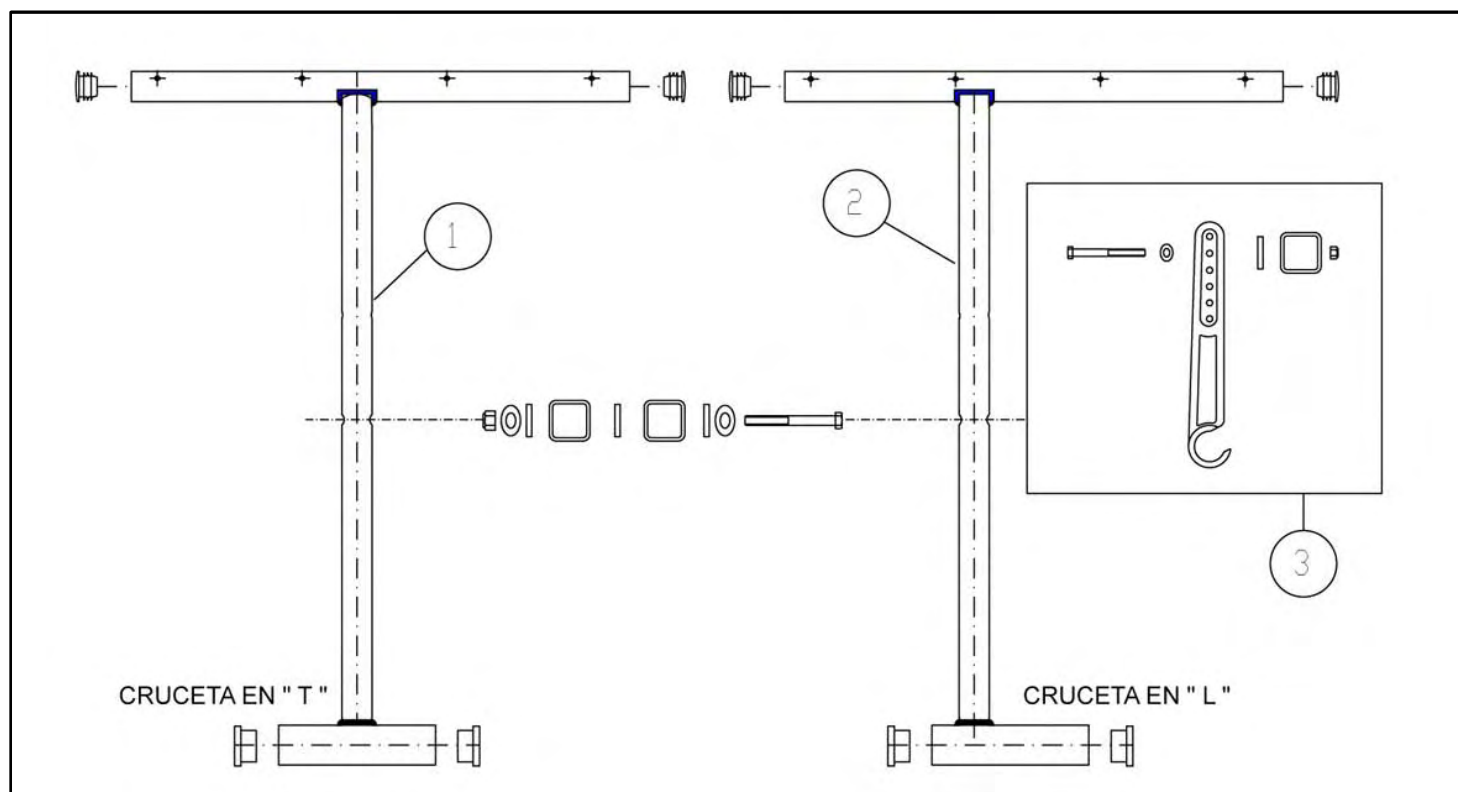

Pos	Item number	Description
1-2	C01789	Cross tube assembly 37 cm
1	C01783N	Cross tube assy. in T. for 37 cm
2	C01777N	Cross tube assy. in L. for 37 cm
1-2	C01790	Cross tube assembly 40 cm
1	C01784N	Cross tube assy. in T. for 40 cm
2	C01778N	Cross tube assy. in L. for 40 cm
1-2	C01791	Cross tube assembly 43 cm
1	C01785N	Cross tube assy. in T. for 43 cm
2	C01779N	Cross tube assy. in L. for 43 cm
1-2	C01792	Cross tube assembly 46 cm
1	C01786N	Cross tube assy. in T. for 46 cm
2	C01780N	Cross tube assy. in L. for 46 cm
1-2	C01793	Cross tube assembly 49 cm
1	C01787N	Cross tube assy. in T. for 49 cm
2	C01781N	Cross tube assy. in L. for 49 cm
1-2	C01794	Cross tube assembly 52 cm
1	C01788N	Cross tube assy. in T. for 52 cm
2	C01782N	Cross tube assy. in L. for 52 cm
3	C011018	X-hinge kit assy.


Breezy 300 - Side frame, up until 07/05/2012 (no. 30012050005)

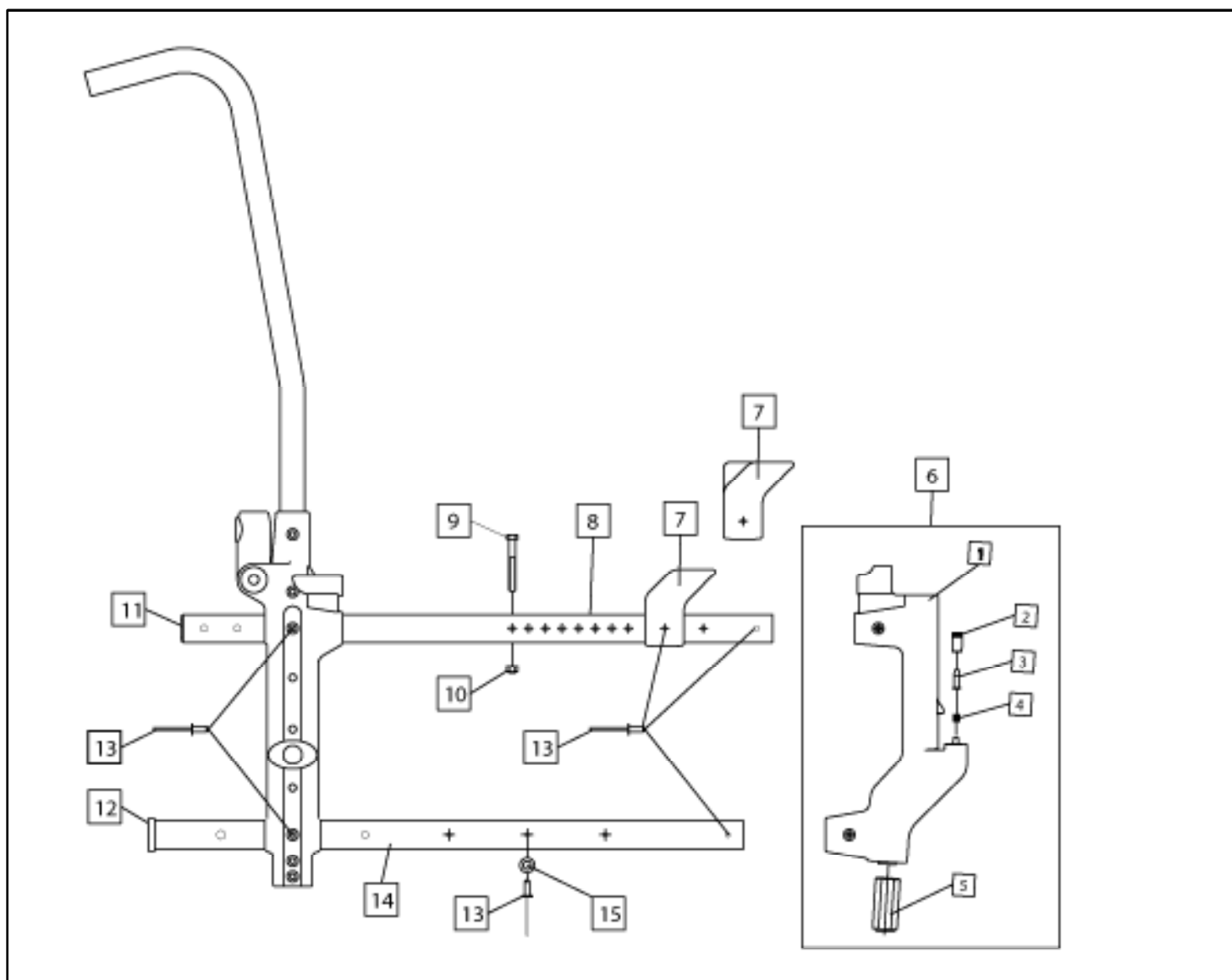

Pos	Item number	Description	Comment
1	P01347	Front vertical	
2	P03077	Footrests Clamping Pin	
3	P03076	Locking nozzle	
4	P15035	Spring	
5	P03158	Bracket fastening socket	
6	C01800	Sub. front vertical	
7	P01201	Armrest receiver - RH before 3/5/06	Up until 3/05/06
7	P01202	Armrest receiver - RL before 3/5/06	Up until 3/05/06
7	P01623	Armrest Receiver with Clamp - RH	From 3/05/06
7	P01624	Armrest Receiver with Clamp - LH	From 3/05/06

Pos	Item number	Description	Comment
8	P11319GS	Upper structure tube, selenium grey	
8	P11319VM	Upper structure tube, apple green	
8	P11319NB	Upper structure tube, brilliant black	
8	P11319RB	Upper structure tube, brilliant red	
9	P04216	Din 912 M.6 x 35 bolt	
10	P04013	Din 985 M.6 nut	
11	P01197	0 25 plug	
12	P01327	0 25 open end	From 1/06/06
13	P04189	04.8 x 11 rivet	
14	P11053GS	Lower structure tube, selenium grey	
14	P11053VM	Lower structure tube, apple green	
14	P11053NB	Lower structure tube, brilliant black	
14	P11053RB	Lower structure tube, brilliant red	
15	P01185	Cross brace end	Up until 1/09/06
15	P02341	Cross brace end	From 1/09/06


Breezy 300 - Side frame from 07/05/2012 (no. 30012050005)

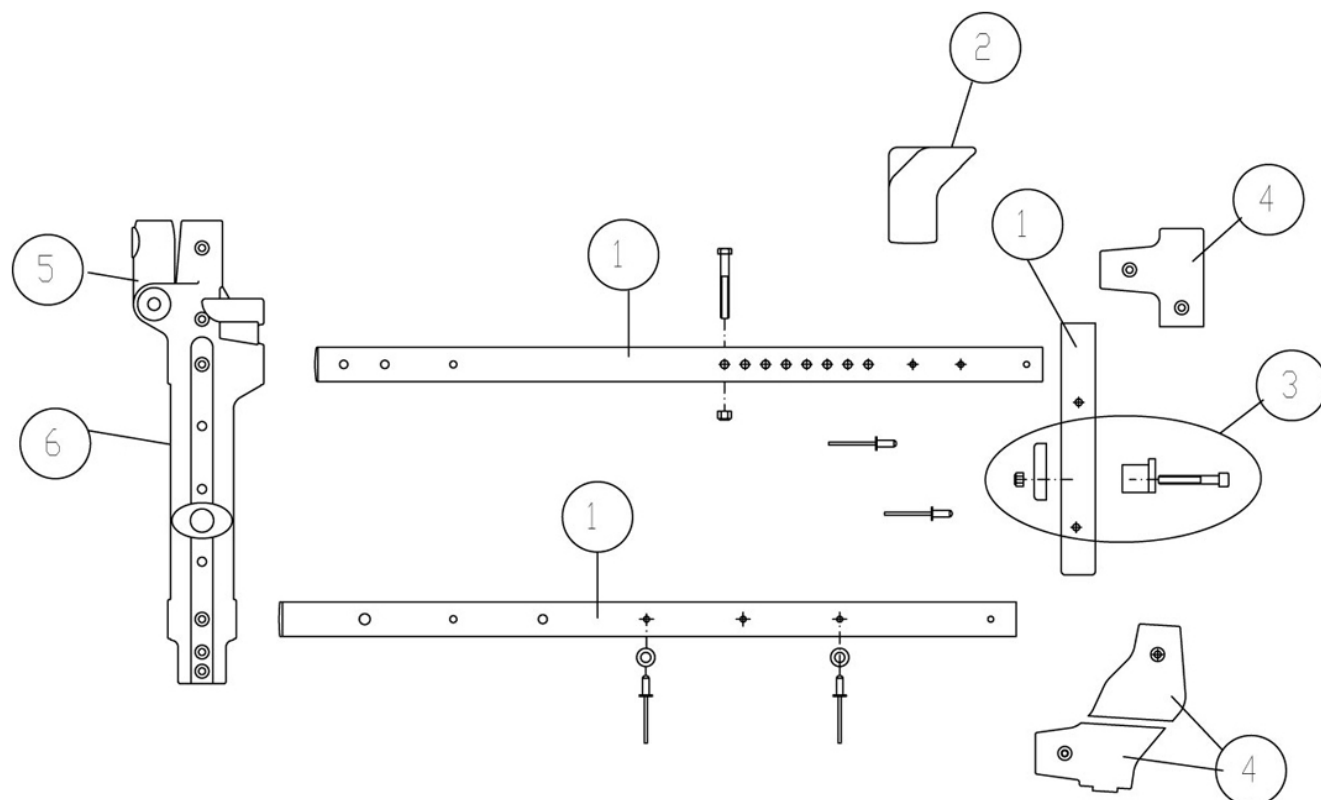

REMACHES CORTOS MENOS EN LAS PIEZAS Nº 4

Pos	Item number	Description
1	C011019GS	Side frame tube, selenium grey
1	C011019VM	Side frame tube, apple green
1	C011019NB	Side frame tube, brilliant black
1	C011019RB	Side frame tube, brilliant red
2	C011020	Armrest front brackets
3	064925	Footrests latch support
4	C04102	Front vertical assemblies
5	P01183	Armrest locking/support clamp assembly
6	P01195	Vertical Assemblies
6	P01352	Quick Release Vertical Assemblies

Breezy 300 - Backrest tubes up until 07/05/2012 (no. 30012050005)

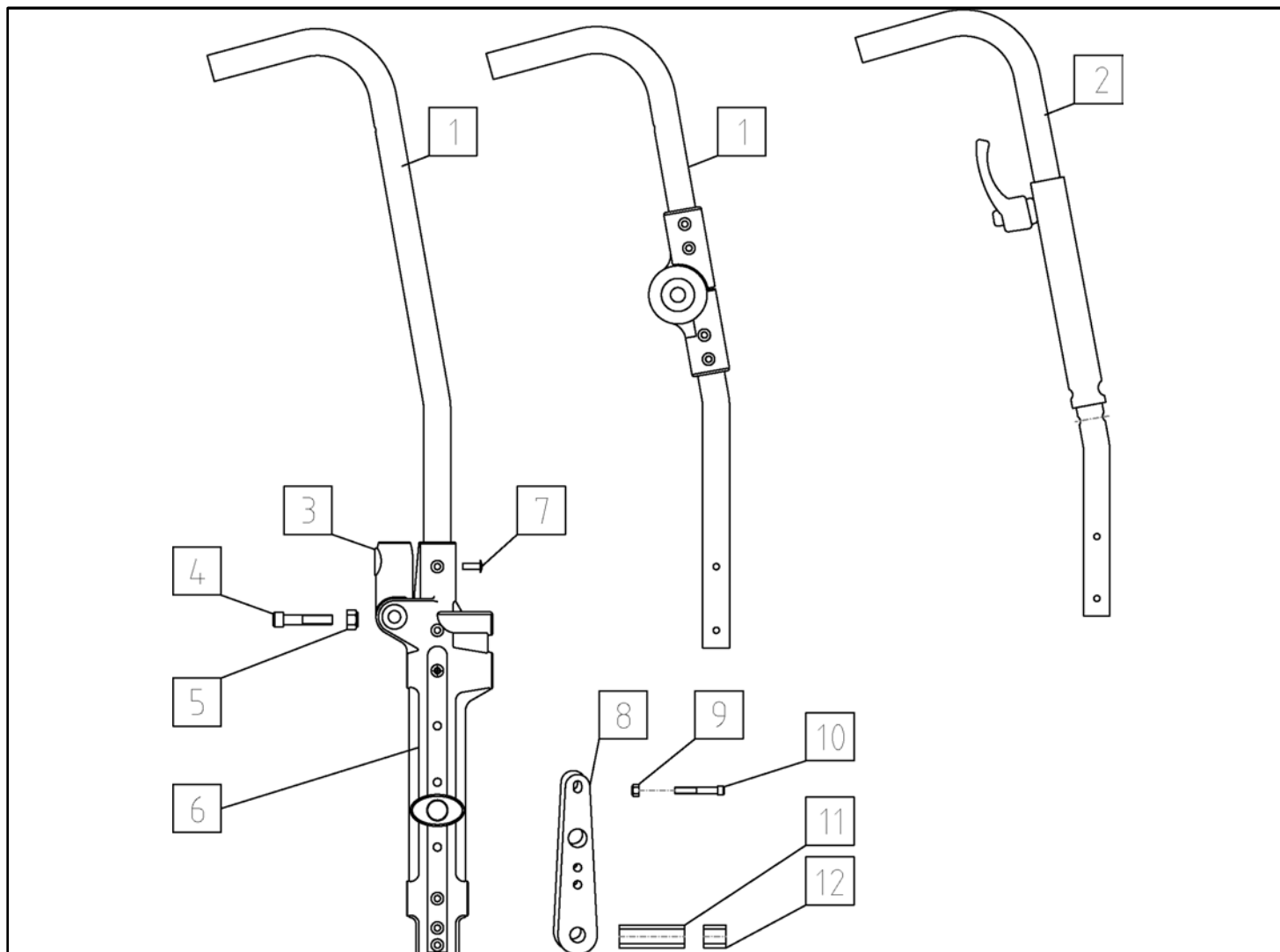


Pos	Item number	Description
1-7	C01351GS	Rear vertical assembly - B305 selenium grey
1-7	C01351VM	Rear vertical assembly - B305 apple green
1-7	C01351NB	Rear vertical assembly - B305 brilliant black
1-7	C01351RB	Rear vertical assembly - B305 brilliant red
1-7	C01350GS	Rear vertical assembly - B312 selenium grey
1-7	C01350VM	Rear vertical assembly - B312 apple green
1-7	C01350NB	Rear vertical assembly - B312 brilliant black
1-7	C01350RB	Rear vertical assembly - B312 brilliant red
1-7	C01416D	Rear vertical assembly - B315 - RH
1-7	C01416L	Rear vertical assembly - B315 - LH
1-7	C01417D	Rear vertical assembly - B310 - RH
1-7	C01417L	Rear vertical assembly - B310 - LH
1	P11062GS	Backrest tube B305 - B312 selenium grey

Breezy 300 - Backrest tubes up until 07/05/2012 (no. 30012050005)


Pos	Item number	Description
1	P11062VM	Backrest tube B305 - B312 apple green
1	P11062NB	Backrest tube B305 - B312 brilliant black
1	P11062RB	Backrest tube B305 - B312 brilliant red
1	C01415D	Backrest tube RH (B310 - B315)
1	C01415I	Backrest tube LH (B310 - B315)
2	C01806	Height adjustable backrest tube (B305 - 312) (not included)
3	P01183	Swivel part
4	P04188	Din 912 M 8 x 40 bolt
5	P04006	Din 985 M 8 nut
6	P01346	Vertical
6	P01353G	Vertical - Quick release (B305 - B315)
7	P04189	4.8 x 11 rivet
8	P02293	Docking (B310 - 312)
9	P04013	Din 985 M 6 nut (B310 - 312)
10	P04135	Din 931 M 6 x 60 bolt (B310 - 312)
11	P03167	Spacer socket (B310 - 312)
12	P03168	Spacer socket (B310 - 312)
	P01024	Handle grip

Breezy 300 - Backrest tubes from 07/05/2012 (no. 30012050005)

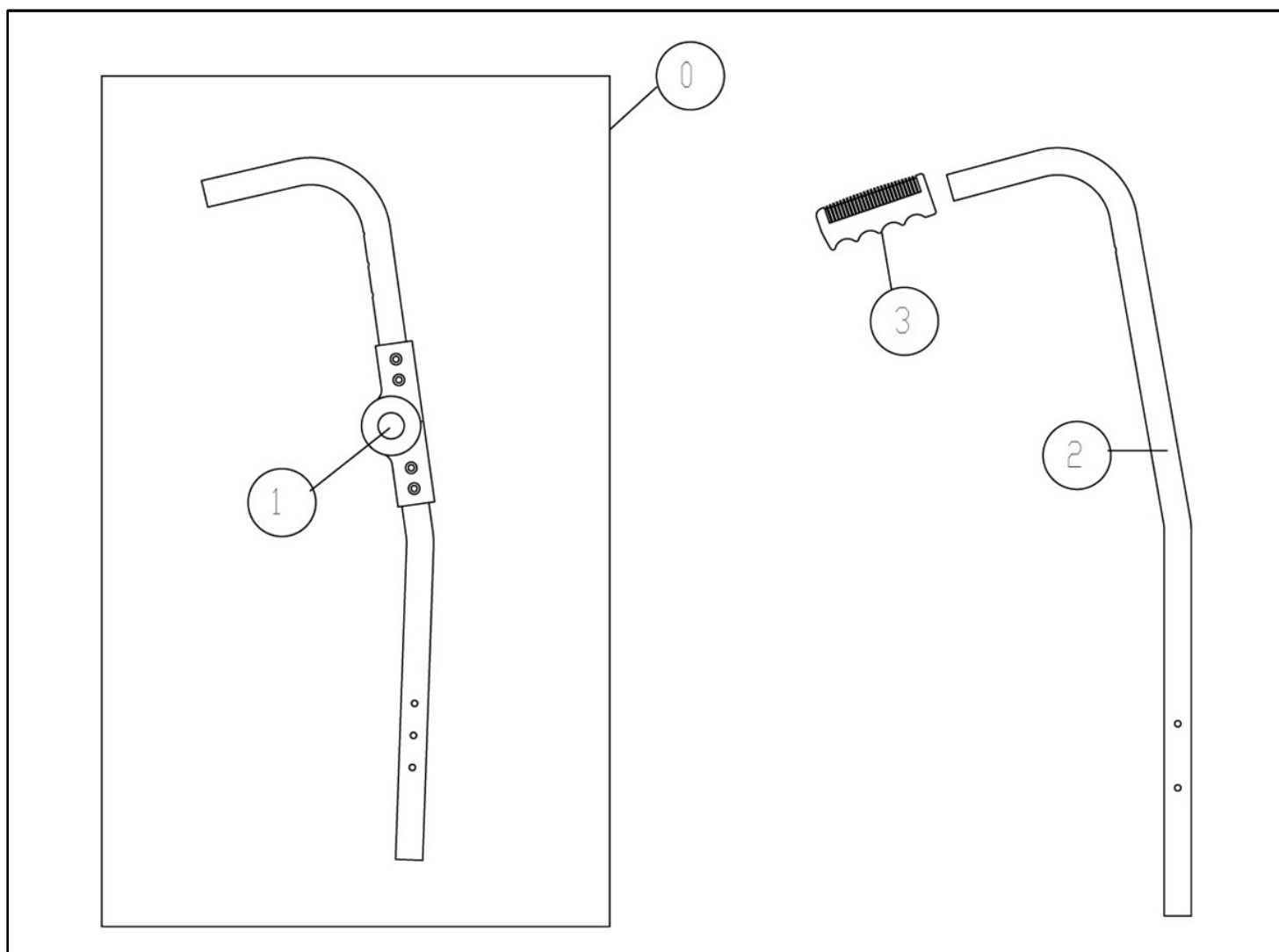


Pos	Item number	Description
0	C01415D	Tubes for Two-part Backrest, right-hand side
0	C01415I	Tubes for Two-part Backrest, left-hand side
1	P01277	Press button
2	P11062GS	Standard backrest tube, selenium grey
2	P11062VM	Standard backrest tube, apple green
2	P11062NB	Standard backrest tube, brilliant black
2	P11062RB	Standard backrest tube, brilliant red
3	P01024	Handle grip

Breezy 300 - Back tube - Reclining model

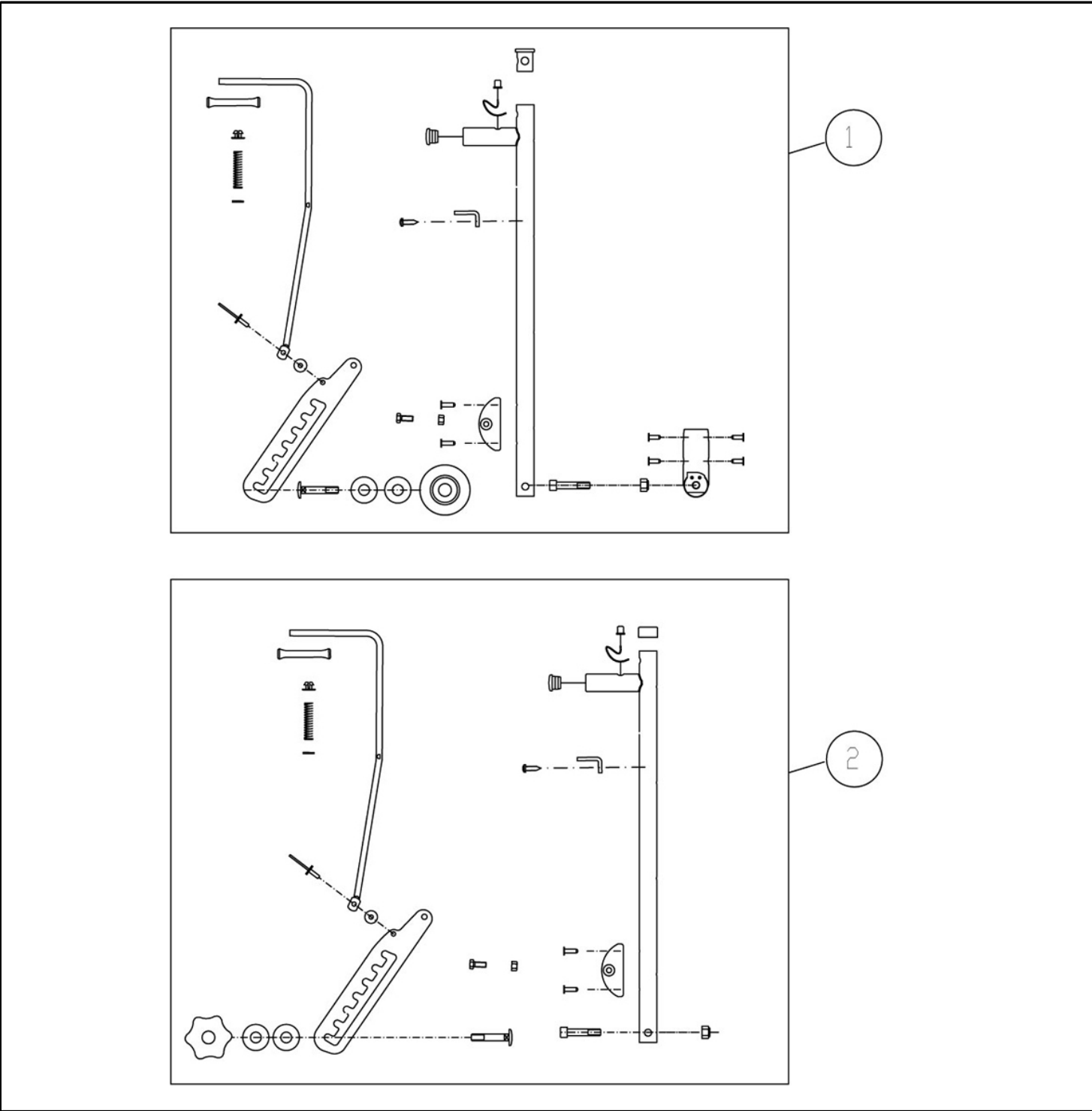


Pos	Item number	Description
1	C011034	Complete reclining backrest tube assy., nylon
2	C011035	Complete reclining backrest tube assy., welded nylon (Skay)

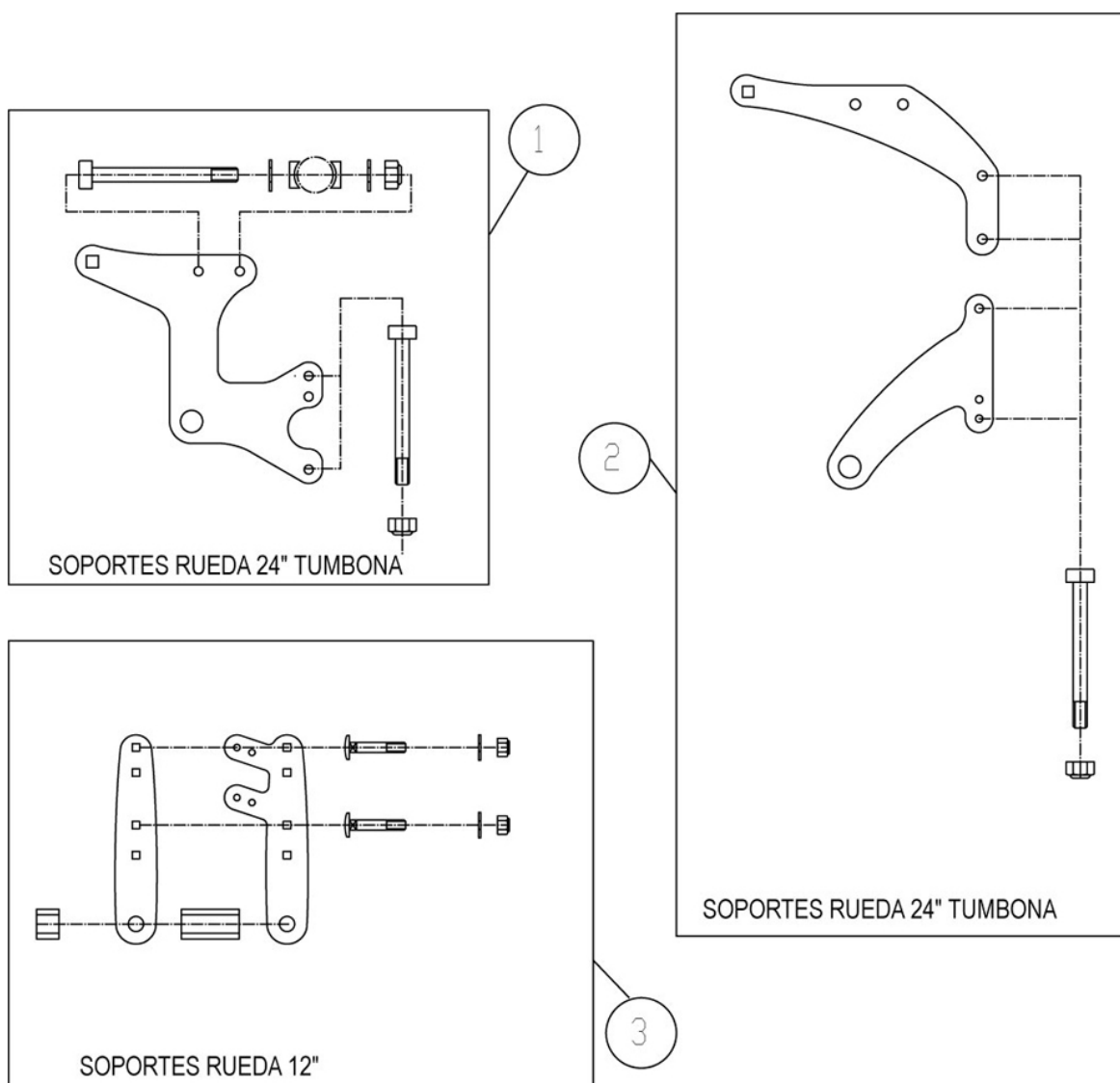Breezy 300 - Axle plate for wheels and backrests


Pos	Item number	Description
1	C011031	24" wheel kit axle plate - Reclining model
2	C011032	12" wheel kit axle plate - Reclining model
3	C011033	12" wheel kit axle plate

Breezy 300 - Seat upholstery

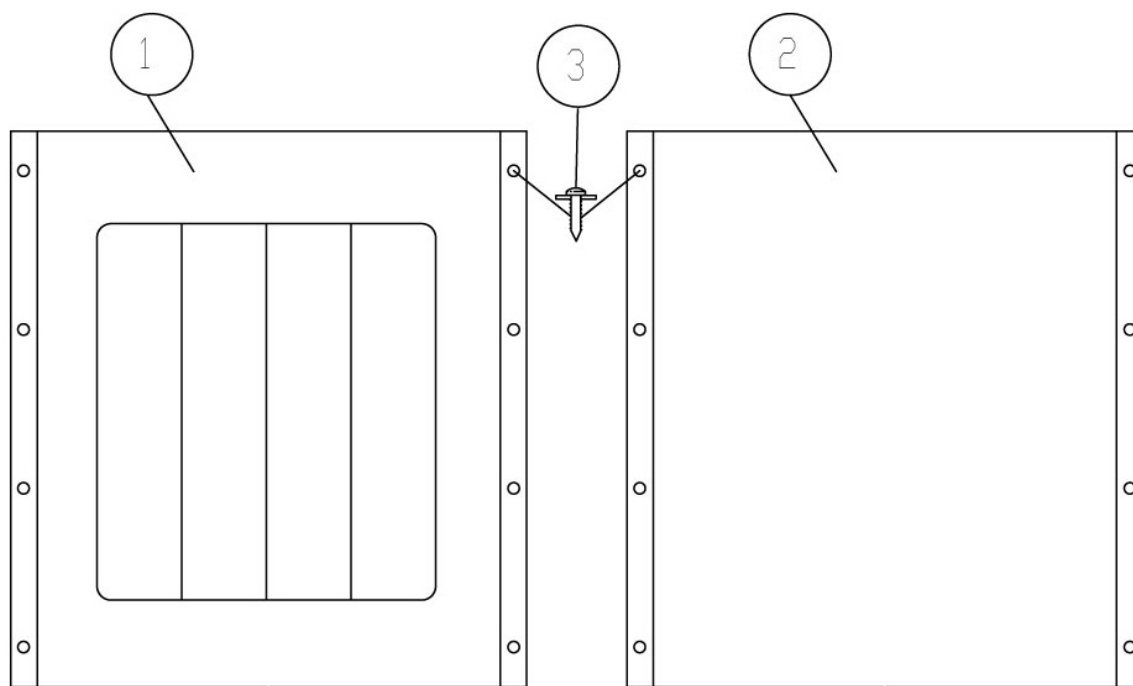


Pos	Item number	Description
1	C02677	Seat upholstery WELDED NYLON (skay) 37 cm, GREY
1	C02678	Seat upholstery WELDED NYLON (skay) 40 cm, GREY
1	C02679	Seat upholstery WELDED NYLON (skay) 43 cm, GREY
1	C02680	Seat upholstery WELDED NYLON (skay) 46 cm, GREY
1	C02681	Seat upholstery WELDED NYLON (skay) 49 cm, GREY
1	C02682	Seat upholstery WELDED NYLON (skay) 52 cm, GREY
2	C02700	Seat upholstery NYLON 37 cm, BLACK
2	C02701	Seat upholstery NYLON 40 cm, BLACK
2	C02702	Seat upholstery NYLON 43 cm, BLACK
2	C02703	Seat upholstery NYLON 46 cm, BLACK
2	C02704	Seat upholstery NYLON 49 cm, BLACK
2	C02705	Seat upholstery NYLON 52 cm, BLACK
2	C02469	Seat upholstery NYLON 37 cm, GREY
2	C02470	Seat upholstery NYLON 40 cm, GREY
2	C02471	Seat upholstery NYLON 43 cm, GREY
2	C02472	Seat upholstery NYLON 46 cm, GREY
2	C02473	Seat upholstery NYLON 49 cm, GREY
2	C02474	Seat upholstery NYLON 52 cm, GREY
3	P04332	Upholstery fixing bolt

Breezy 300 - Anatomical seat

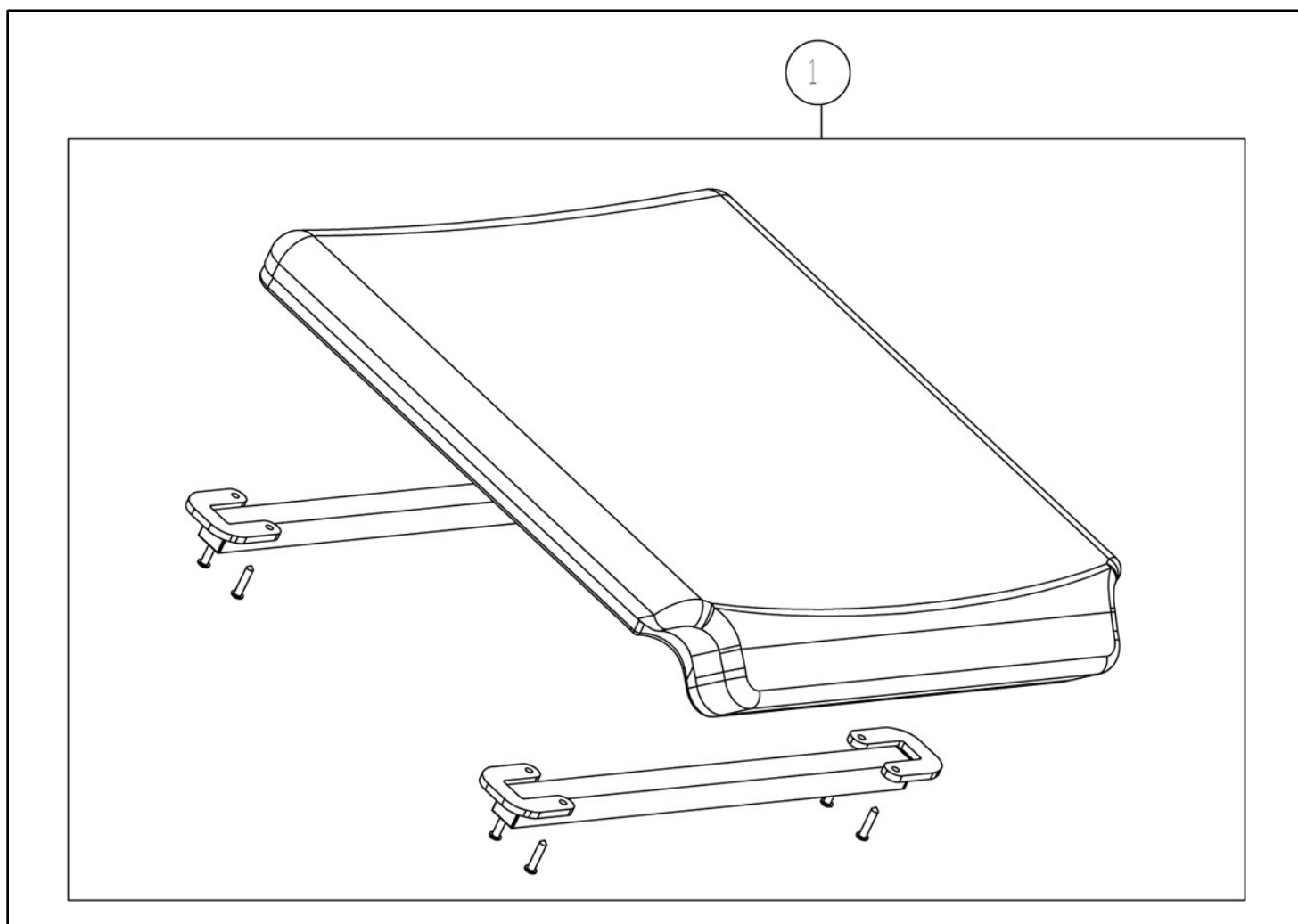


Pos	Item number	Description
1	C02580	Anatomic seat assembly 37 cm
1	C02581	Anatomic seat assembly 40 cm
1	C02582	Anatomic seat assembly 43 cm
1	C02583	Anatomic seat assembly 46 cm
1	C02584	Anatomic seat assembly 49 cm
1	C02585	Anatomic seat assembly 52 cm

Breezy 300 - Anatomical seat with toilet bucket

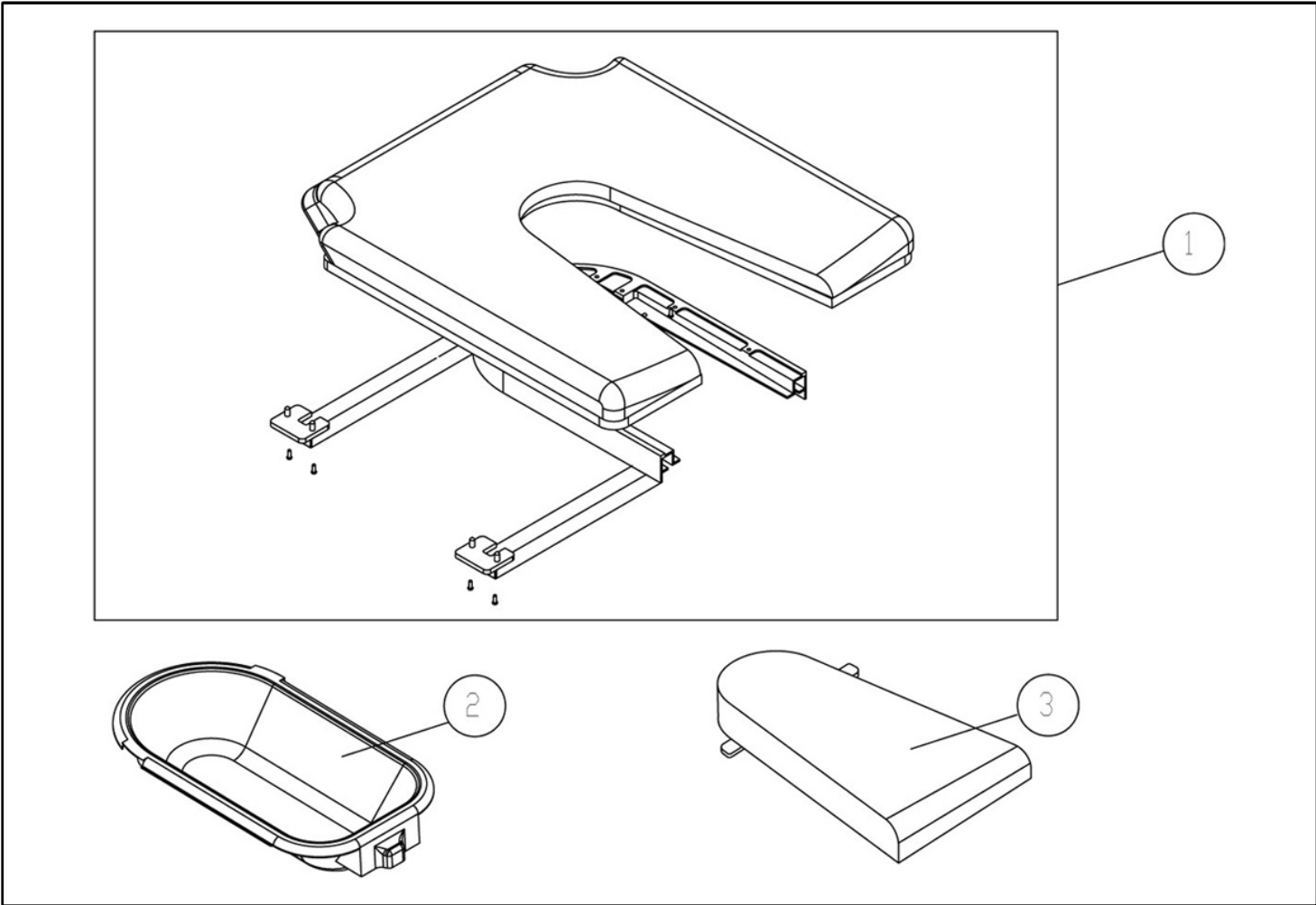

Pos	Item number	Description
1	C02590	Anatomical seat with toilet bucket 37 cm
1	C02591	Anatomical seat with toilet bucket 40 cm
1	C02592	Anatomical seat with toilet bucket 43 cm
1	C02593	Anatomical seat with toilet bucket 46 cm
1	C02594	Anatomical seat with toilet bucket 49 cm
1	C02595	Anatomical seat with toilet bucket 52 cm
2	P01035	Bucket
3	C02597	Bucket lid

Breezy 300 - Backrest Upholstery

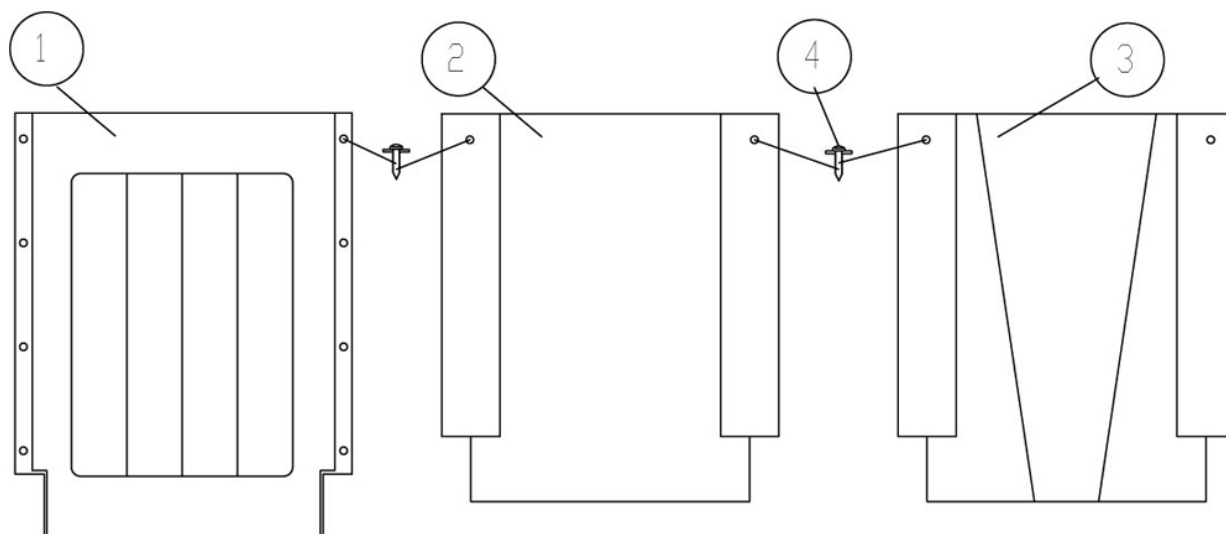


Pos	Item number	Description
1	C02653	Backrest upholstery WELDED NYLON (skay) 37 cm, GREY
1	C02654	Backrest upholstery WELDED NYLON (skay) 40 cm, GREY
1	C02655	Backrest upholstery WELDED NYLON (skay) 43 cm, GREY
1	C02656	Backrest upholstery WELDED NYLON (skay) 46 cm, GREY
1	C02657	Backrest upholstery WELDED NYLON (skay) 49 cm, GREY
1	C02658	Backrest upholstery WELDED NYLON (skay) 52 cm, GREY
2	C02706	Backrest upholstery NYLON 37 cm, BLACK
2	C02707	Backrest upholstery NYLON 40 cm, BLACK
2	C02708	Backrest upholstery NYLON 43 cm, BLACK
2	C02709	Backrest upholstery NYLON 46 cm, BLACK
2	C02710	Backrest upholstery NYLON 49 cm, BLACK
2	C02711	Backrest upholstery NYLON 52 cm, BLACK
2	C02475	Backrest upholstery NYLON 37 cm, GREY
2	C02476	Backrest upholstery NYLON 40 cm, GREY
2	C02477	Backrest upholstery NYLON 43 cm, GREY
2	C02478	Backrest upholstery NYLON 46 cm, GREY
2	C02479	Backrest upholstery NYLON 49 cm, GREY
2	C02480	Backrest upholstery NYLON 52 cm, GREY
3	C02730	Breathable vented backrest upholstery, 37 cm
3	C02731	Breathable vented backrest upholstery, 40 cm
3	C02732	Breathable vented backrest upholstery, 43 cm
3	C02733	Breathable vented backrest upholstery, 46 cm
3	C02734	Breathable vented backrest upholstery, 49 cm

Breezy 300 - Backrest Upholstery


Pos	Item number	Description
3	C02735	Breathable vented backrest upholstery, 52 cm
4	P04332	Upholstery fixing bolt

Breezy 300 - Two-Part Backrest upholstery

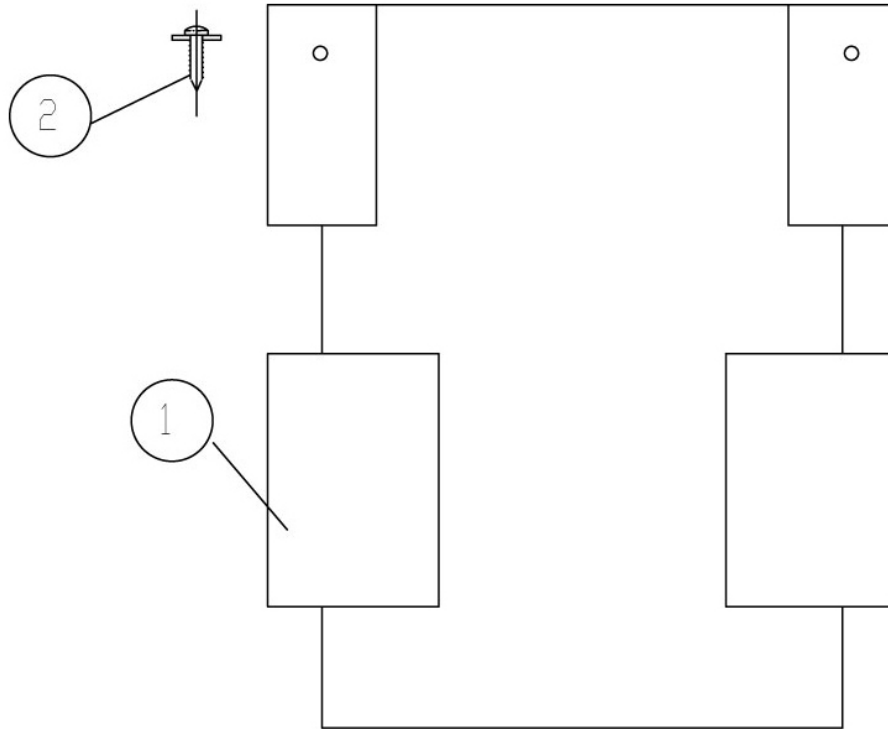

Pos	Item number	Description
1	C02712	Half folding backrest upholstery NYLON 37 cm, BLACK
1	C02713	Half folding backrest upholstery NYLON 40 cm, BLACK
1	C02714	Half folding backrest upholstery NYLON 43 cm, BLACK
1	C02715	Half folding backrest upholstery NYLON 46 cm, BLACK
1	C02716	Half folding backrest upholstery NYLON 49 cm, BLACK
1	C02717	Half folding backrest upholstery NYLON 52 cm, BLACK
1	C02481	Half folding backrest upholstery NYLON 37 cm, GREY
1	C02482	Half folding backrest upholstery NYLON 40 cm, GREY
1	C02483	Half folding backrest upholstery NYLON 43 cm, GREY
1	C02484	Half folding backrest upholstery NYLON 46 cm, GREY
1	C02485	Half folding backrest upholstery NYLON 49 cm, GREY
1	C02486	Half folding backrest upholstery NYLON 52 cm, GREY
2	P04332	Upholstery fixing bolt


Breezy 300 - Reclining Backrest Upholstery

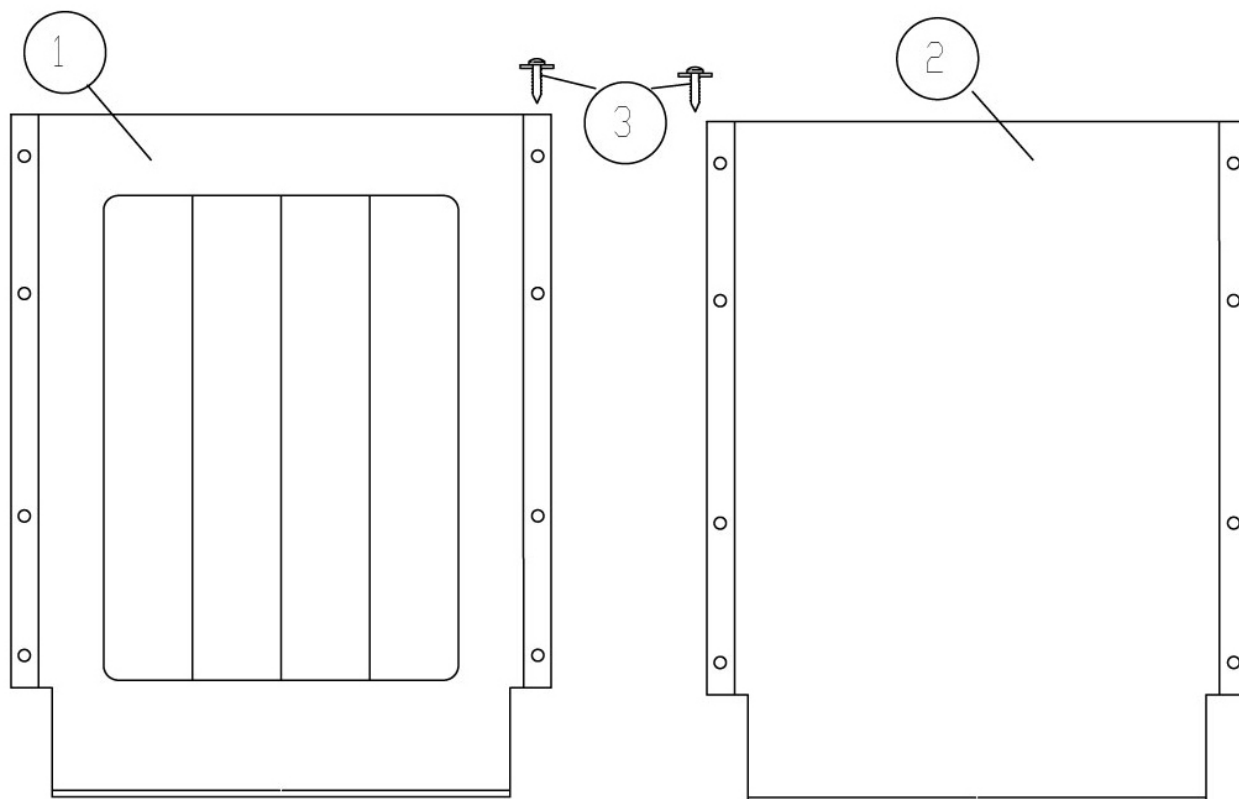

Pos	Item number	Description
1	C02665	Reclining backrest upholstery WELDED NYLON (skay) 37 cm
1	C02666	Reclining backrest upholstery WELDED NYLON (skay) 40 cm
1	C02667	Reclining backrest upholstery WELDED NYLON (skay) 43 cm
1	C02668	Reclining backrest upholstery WELDED NYLON (skay) 46 cm
1	C02669	Reclining backrest upholstery WELDED NYLON (skay) 49 cm
1	C02670	Reclining backrest upholstery WELDED NYLON (skay) 52 cm
2	C02718	Reclining backrest upholstery NYLON 37cm, BLACK
2	C02719	Reclining backrest upholstery NYLON 40 cm, BLACK
2	C02720	Reclining backrest upholstery NYLON 43 cm, BLACK
2	C02721	Reclining backrest upholstery NYLON 46 cm, BLACK
2	C02722	Reclining backrest upholstery NYLON 49 cm, BLACK
2	C02723	Reclining backrest upholstery NYLON 52 cm, BLACK
2	C02487	Reclining backrest upholstery NYLON 37 cm, GREY
2	C02488	Reclining backrest upholstery NYLON 40 cm, GREY
2	C02489	Reclining backrest upholstery NYLON 43 cm, GREY
2	C02490	Reclining backrest upholstery NYLON 46 cm, GREY
2	C02491	Reclining backrest upholstery NYLON 49 cm, GREY


Pos	Item number	Description
2	C02492	Reclining backrest upholstery NYLON 52 cm, GREY
3	P04332	Upholstery fixing bolt

Breezy 300 - Anatomical backrest

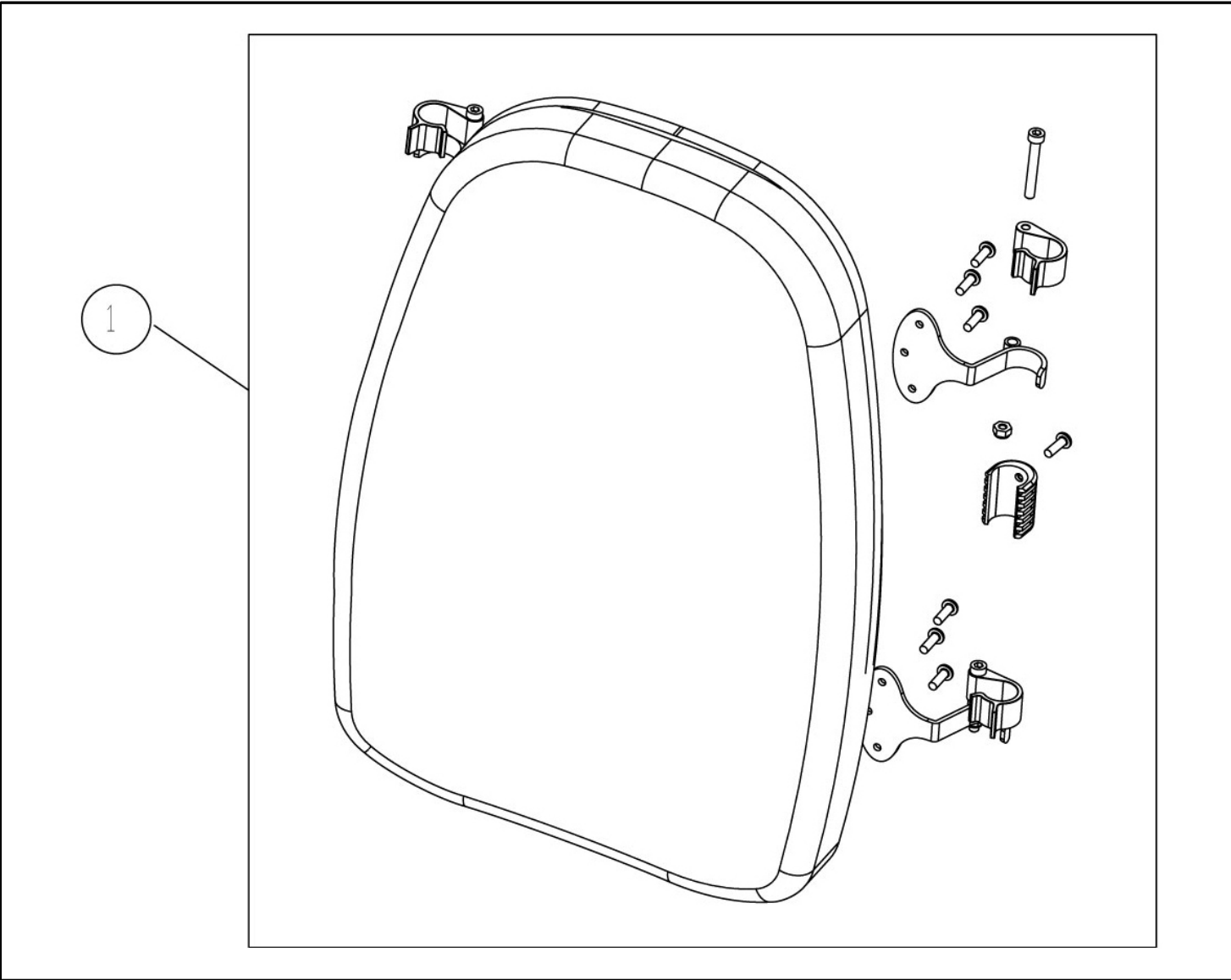


Pos	Item number	Description
1	C02600	Anatomical backrest 37 cm
1	C02601	Anatomical backrest 40 cm
1	C02602	Anatomical backrest 43 cm
1	C02603	Anatomical backrest 46 cm
1	C02604	Anatomical backrest 49 cm
1	C02605	Anatomical backrest 52 cm

Breezy 300 - Stiffener tube

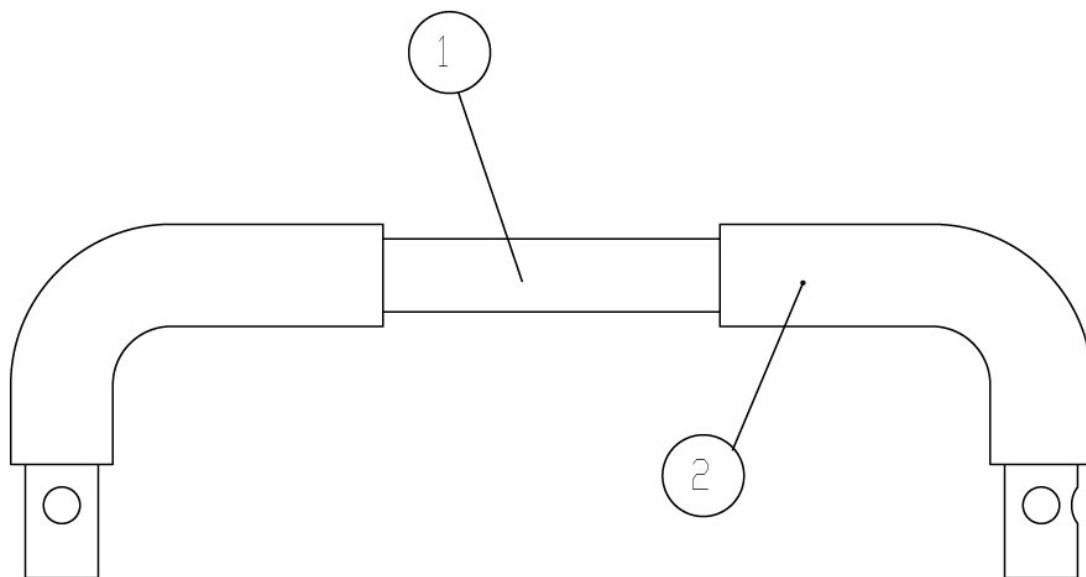


Pos	Item number	Description	Comment
s/n	C01705	Complete Stiffener tube assembly 37 cm	
s/n	C01706	Complete Stiffener tube assembly 40 cm	
s/n	C01707	Complete Stiffener tube assembly 43 cm	
s/n	C01708	Complete Stiffener tube assembly 46 cm	
s/n	C01709	Complete Stiffener tube assembly 49 cm	
s/n	C01710	Complete Stiffener tube assembly 52 cm	
1	P11303N	37 cm tube	
1	P11304N	40 cm tube	
1	P11305N	43 cm tube	
1	P11306N	46 cm tube	
1	P11307N	49 cm tube	
1	P11308N	52 cm tube	
2	P09053	G.E Tubular 32 x 22 x 220	2 pcs.

Breezy 300 - Height-adjustable push handles

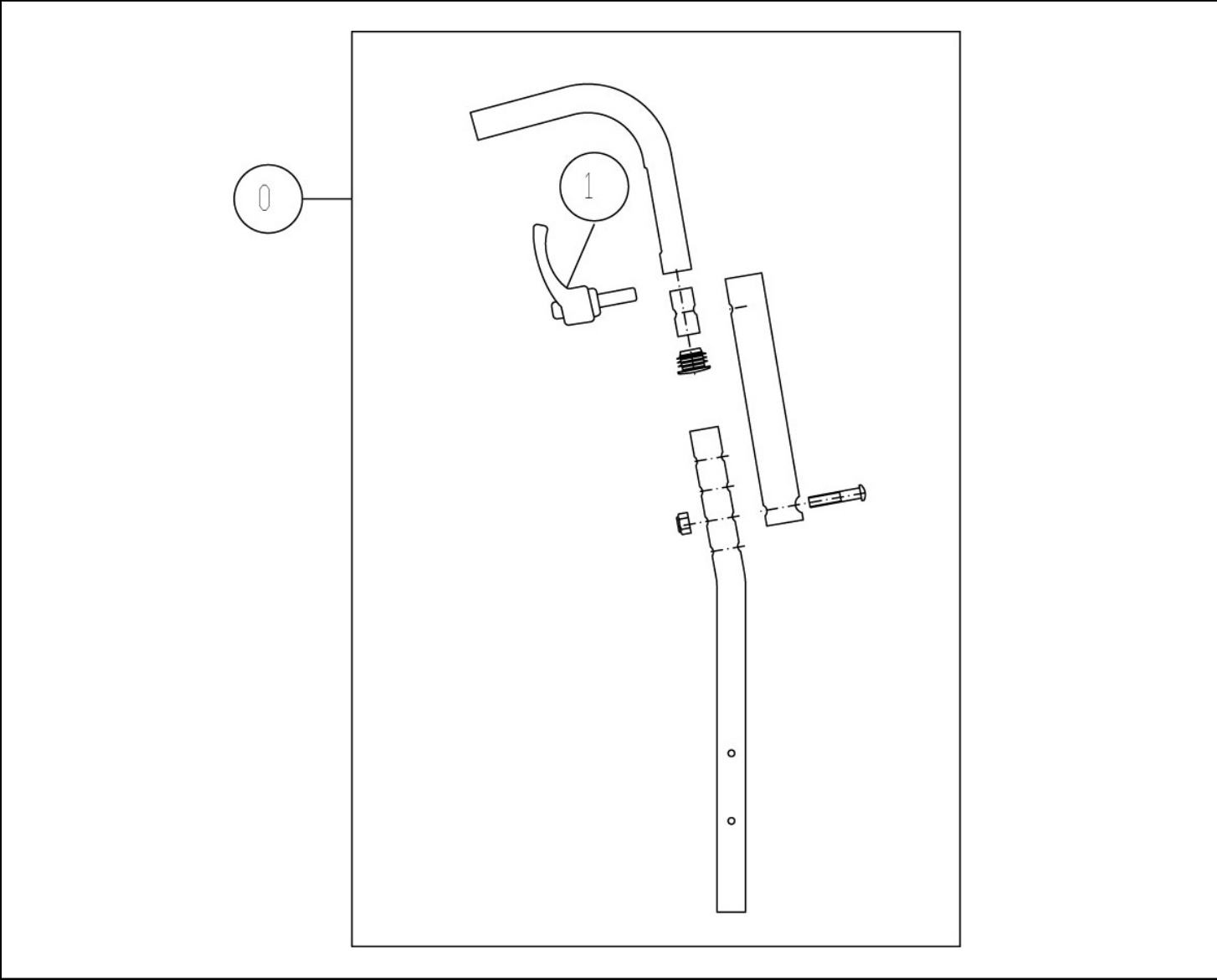

Pos	Item number	Description
0	C01806	Complete height adjustable backrest tube assy.
1	P01641	Adjustable push handle M6x25, black

Breezy 300 - Armrests

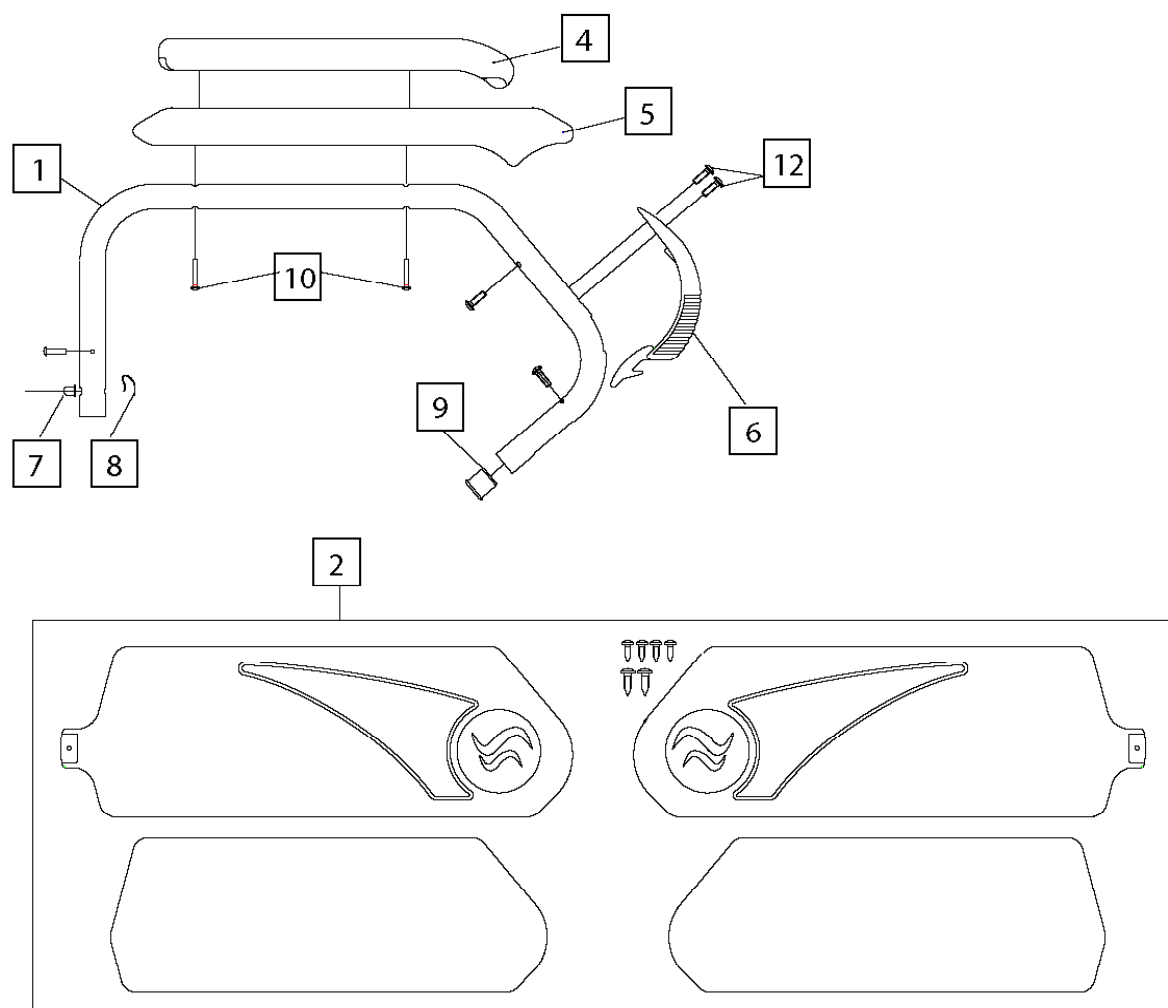


Pos	Item number	Description	Comment
1-12	C07084VM	Complete armrest, apple green - Right	We recommend that you also replace left hand
1-12	C07084NB	Complete armrest, brilliant black - Right	We recommend that you also replace left hand
1-12	C07084GS	Complete armrest, selenium grey - Right	We recommend that you also replace left hand
1-12	C07084RB	Complete armrest, brilliant red - Right	We recommend that you also replace left hand

Breezy 300 - Armrests

Pos	Item number	Description	Comment
1-12	C07085VM	Complete armrest, apple green - Left	We recommend that you also replace right hand
1-12	C07085NB	Complete armrest, brilliant black - Left	We recommend that you also replace right hand
1-12	C07085GS	Complete armrest, selenium grey - Left	We recommend that you also replace right hand
1-12	C07085RB	Complete armrest, brilliant red - Left	We recommend that you also replace right hand
1-12	C07090NB	Complete long armrest, brilliant black - Right	We recommend that you also replace left hand
1-12	C07090VM	Complete long armrest, apple green - Right	We recommend that you also replace left hand
1-12	C07090GS	Complete long armrest, selenium grey - Right	We recommend that you also replace left hand
1-12	C07090RB	Complete long armrest, brilliant red - Right	We recommend that you also replace left hand
1-12	C07091VM	Complete long armrest, apple green - Left	We recommend that you also replace right hand
1-12	C07091NB	Complete long armrest, brilliant black - Left	We recommend that you also replace right hand
1-12	C07091GS	Complete long armrest, selenium grey - Left	We recommend that you also replace right hand
1-12	C07091RB	Complete long armrest, brilliant red - Left	We recommend that you also replace right hand
1	P11064GS	Right tube, selenium grey	

Pos	Item number	Description	Comment
1	P11064VM	Right tube, apple green	
1	P11064NB	Right tube, brilliant black	
1	P11064RB	Right tube, brilliant red	
1	P11065GS	Left tube, selenium grey	
1	P11065VM	Left tube, apple green	
1	P11065NB	Left tube, brilliant black	
1	P11065RB	Left tube, brilliant red	
2	C07581	Sideguard RH and LH, complete kit (included with fixing kit)	
4	C02191D	Standard upholstery - RH	
4	C02191I	Standard upholstery - LH	
5	C02212	Long upholstery - RH	
5	C02213	Long upholstery - LH	
6	P01203	Latch Lever	
7	P03062	Locking nozzle	
8	P15030	Spring	
9	P01339	22x 1.5 plug	
10	P04080	966 5x30 bolt for standard upholstery	
10	P04207	966 5X35 bolt for long upholstery	
12	P04234	7504 3.5 x 13	


Breezy 300 - Armrests for reclining backrest

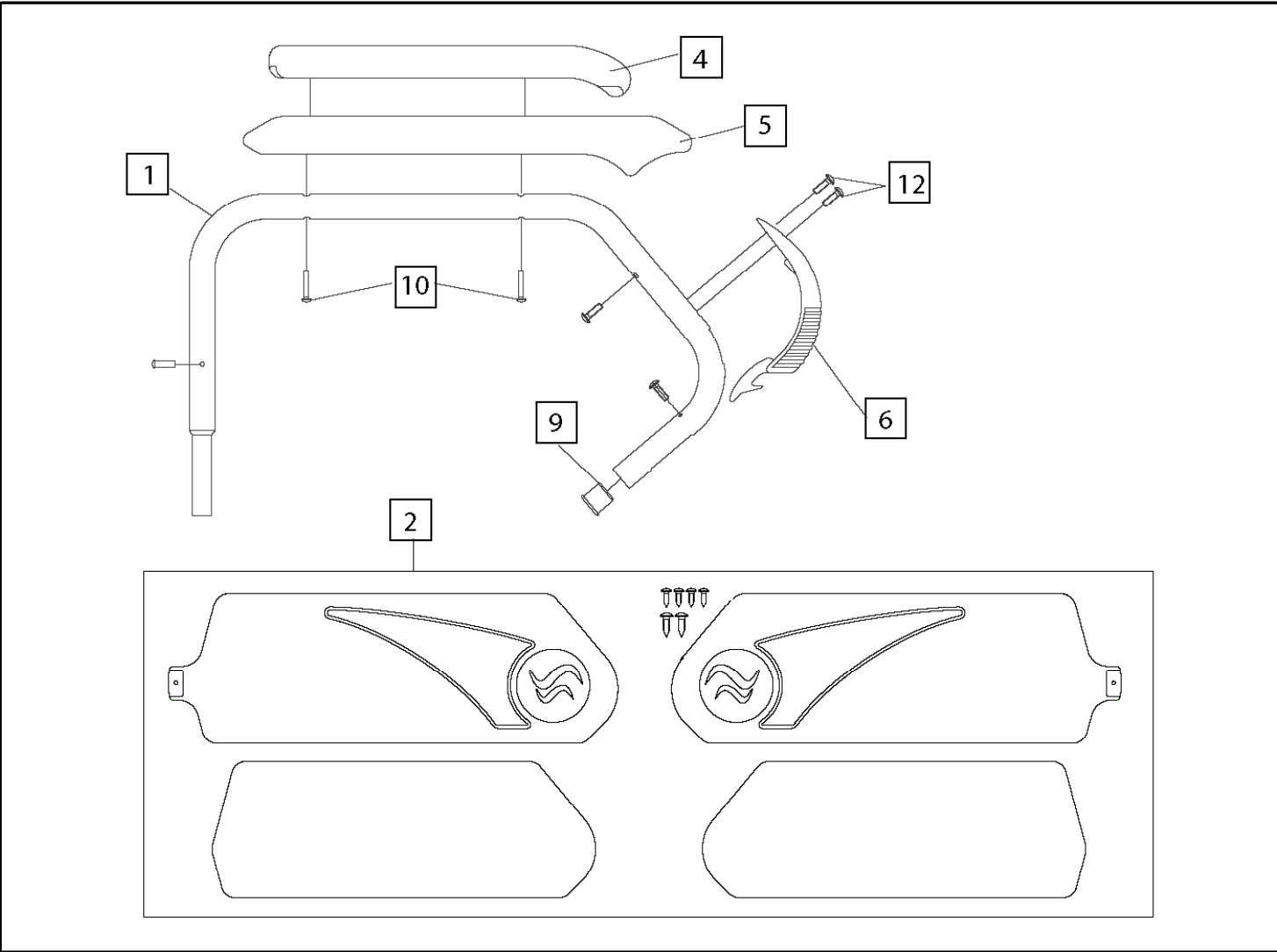

Pos	Item number	Description	Comment
1-12	C07092N	Armrests complete - Right 321-341	We recommend that you also replace left hand
1-12	C07093N	Armrests complete - Left 321-341	We recommend that you also replace right hand
1-12	C07528N	Armrests complete, long - Right 321-341	We recommend that you also replace left hand
1-12	C07529N	Armrests complete, long - Left 321-341	We recommend that you also replace right hand


Pos	Item number	Description	Comment
1	P05621N	Tube - Right	
1	P05622N	Tube - Left	
2	C07583	Sideguard RH and LH, complete kit (included with fixing kit)	
4	C02191D	Standard upholstery - RH	
4	C02191I	Standard upholstery - LH	
5	C02212	Long upholstery - RH	
5	C02213	Long upholstery - LH	
6	P01203	Latch Lever	
9	P01340	22x1 plug	
10	P04080	966 5x30 bolt for standard upholstery	
11	P04207	966 5X35 bolt for long upholstery	
12	P04234	7504 3.5x13	

Breezy 300 - Height adjustable armrests

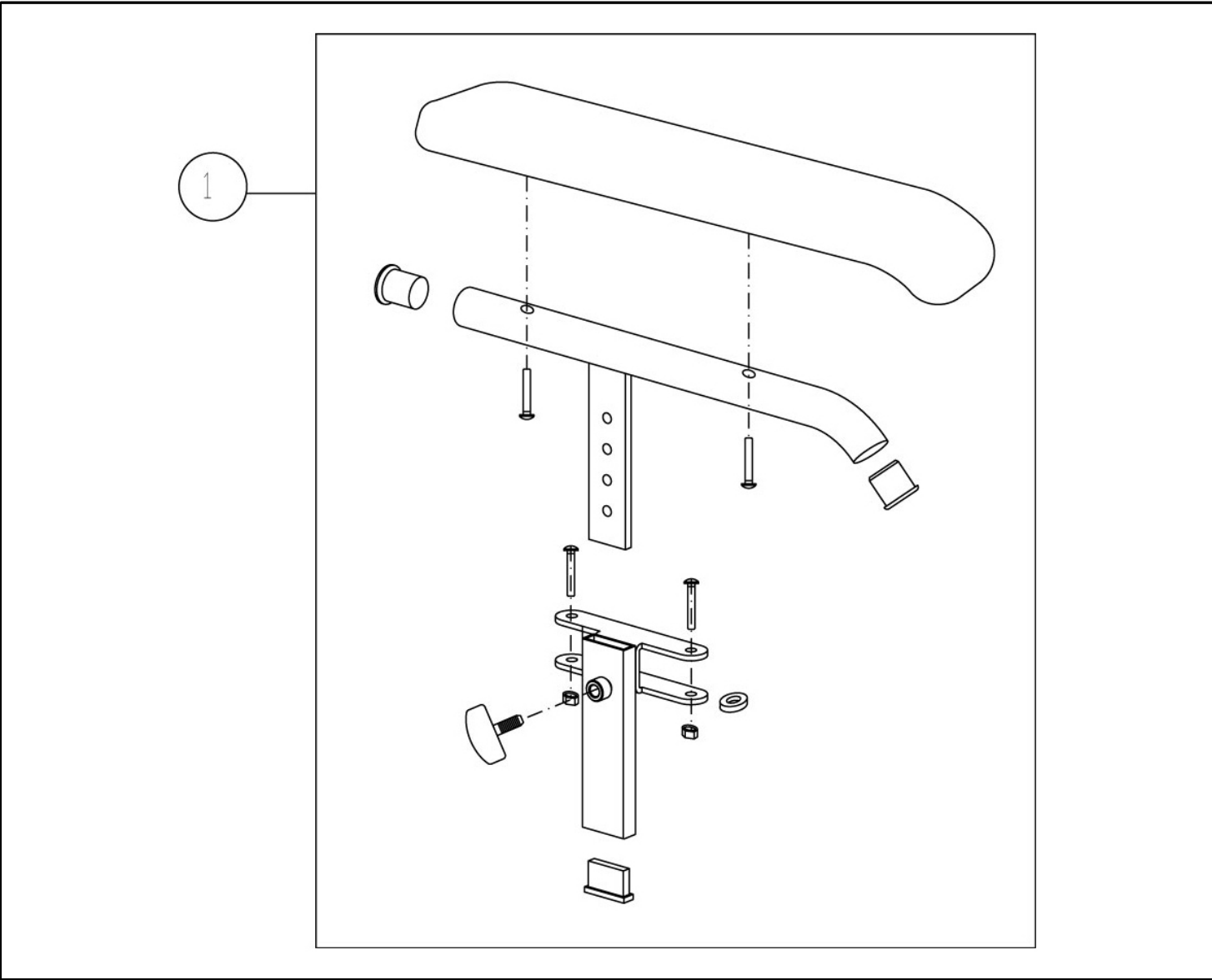

Pos	Item number	Description
1	C07520	Adjustable armrest assembly, complete, right
1	C07519	Adjustable armrest assembly, complete, left

Breezy 300 - Footrests up until 07/05/2012 (no. 30012050005)

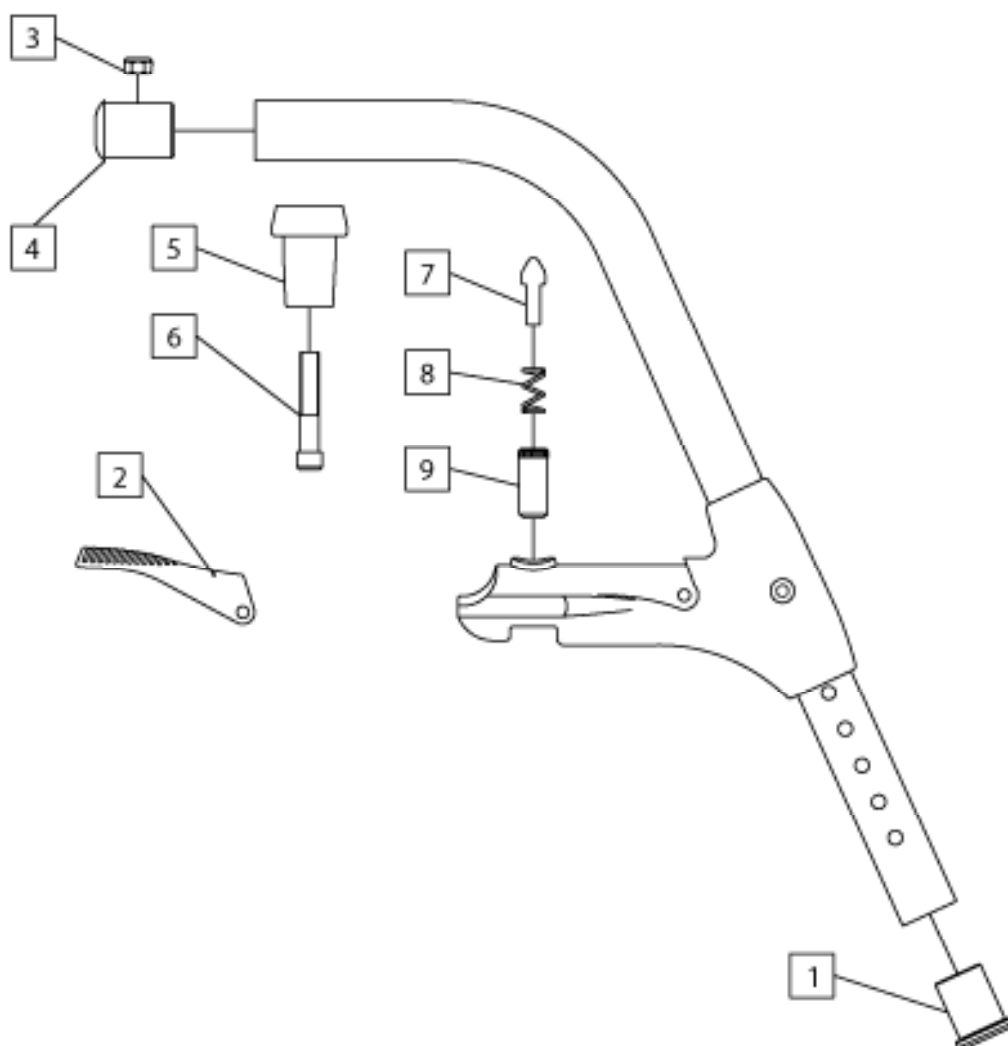


Pos	Item number	Description	Comment
1-9	C05254S	Footrest Assembly Silver Texture OLD	Up until 8/06/09
1-9	C05254AB	Footrest Assembly Brilliant Blue OLD	Up until 8/06/09
1-9	C05254GB	Footrest Assembly Silver OLD	Up until 8/06/09
1-9	C05254GS	Footrests, selenium grey	From 8/06/09
1-9	C05254VM	Footrests, apple green	From 8/06/09
1-9	C05254NB	Footrests, brilliant black	From 8/06/09
1-9	C05254RB	Footrests, brilliant red	
1	P01189	Open end	
2	P01198	Handle cover	

Breezy 300 - Footrests up until 07/05/2012 (no. 30012050005)


Pos	Item number	Description	Comment
3	P04006	Din 985 M8 nut	Up until 1/09/05
4	P01190	Upper plug	
5	P01182	Upper pin	
6	P04195	Din 912 M 8 x 45 bolt	Up until 1/09/05
6	P04338	CL 81 z 6 x 45 bolt	From 1/09/05
7	P01191	Unlocking pivot	
8	P15035	Spring	
9	P03075	Socket	

Breezy 300 - Footrests from 07/05/2012 (no. 30012050005)

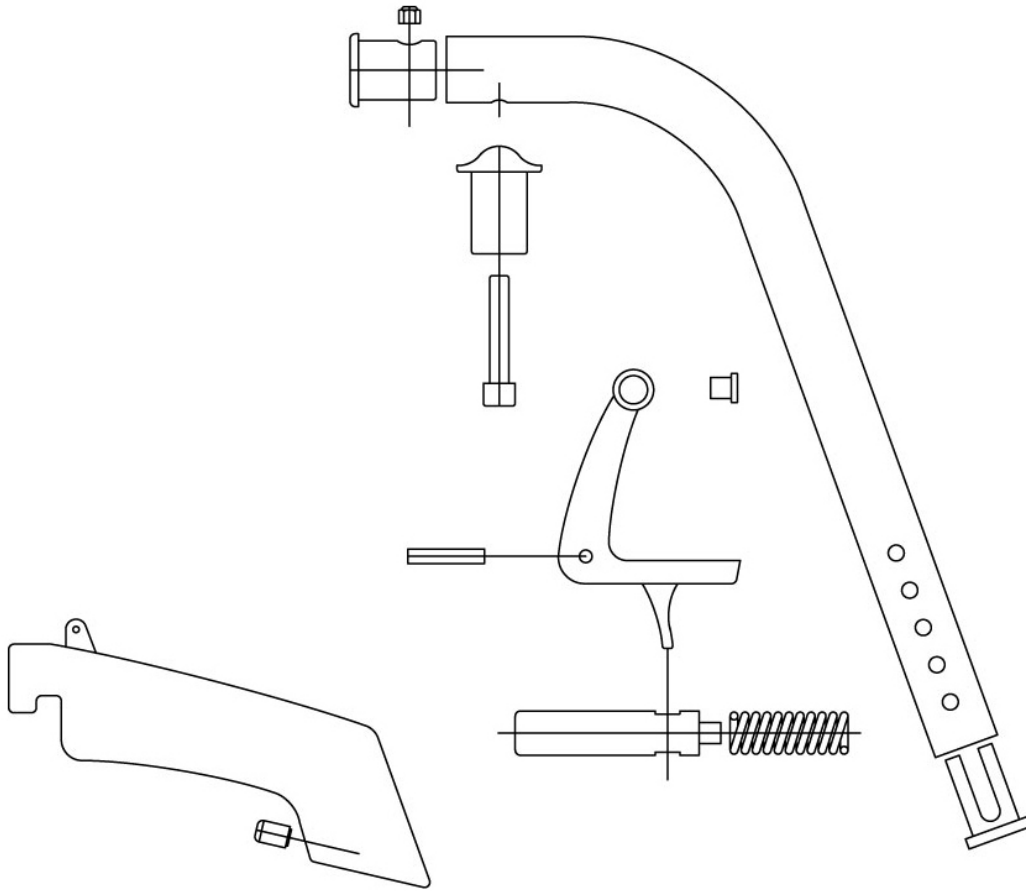


Pos	Item number	Description
s/n	067207-060	Footrests, pair SELENIUM GREY, complete assembly
s/n	067207-061	Footrests, left SELENIUM GREY, complete assembly
s/n	067207-062	Footrests, right SELENIUM GREY, complete assembly
s/n	067207-080	Footrests, pair BRILLIANT RED, complete assembly
s/n	067207-081	Footrests, left BRILLIANT RED, complete assembly
s/n	067207-082	Footrests, right BRILLIANT RED, complete assembly
s/n	067207-090	Footrests, pair APPLE GREEN, complete assembly
s/n	067207-091	Footrests, left APPLE GREEN, complete assembly
s/n	067207-092	Footrests, right VERDE MANZANA, complete assembly
s/n	067207-070	Footrests, pair BRILLIANT BLACK, complete assembly
s/n	067207-071	Footrests, left BRILLIANT BLACK, complete assembly
s/n	067207-072	Footrests, right BRILLIANT BLACK, complete assembly

Breezy 300 - Short footrests

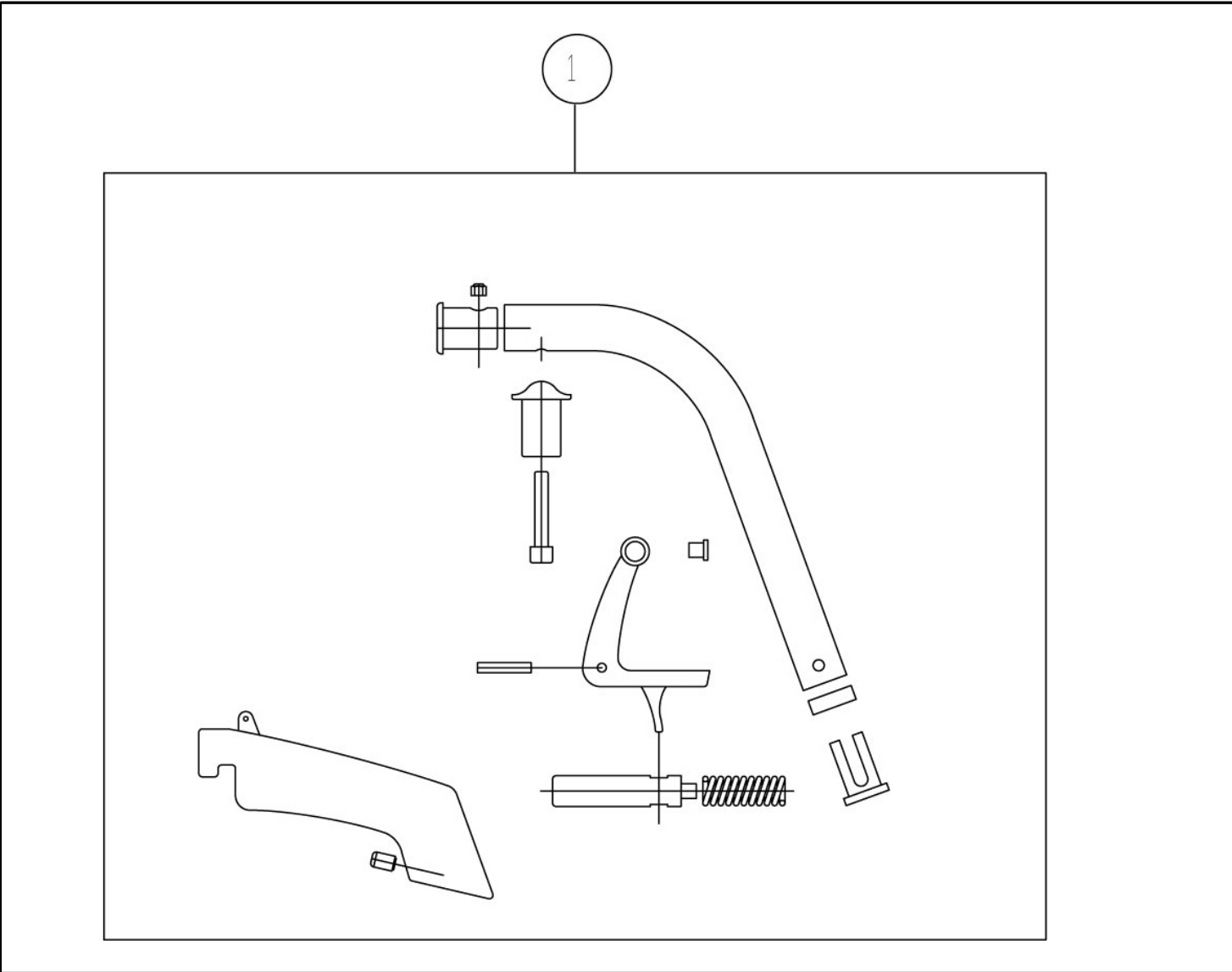

Pos	Item number	Description
1	C05346GS	Short footrests, pair SELENIUM GREY, complete assembly
1	C05346RB	Short footrests, pair BRILLIANT RED, complete assembly
1	C05346VM	Short footrests, pair APPLE GREEN, complete assembly
1	C05346NB	Short footrests, pair BRILLIANT BLACK, complete assembly

Breezy 300 - Elevating legrests up until 07/05/2012 (no. 30012050005)

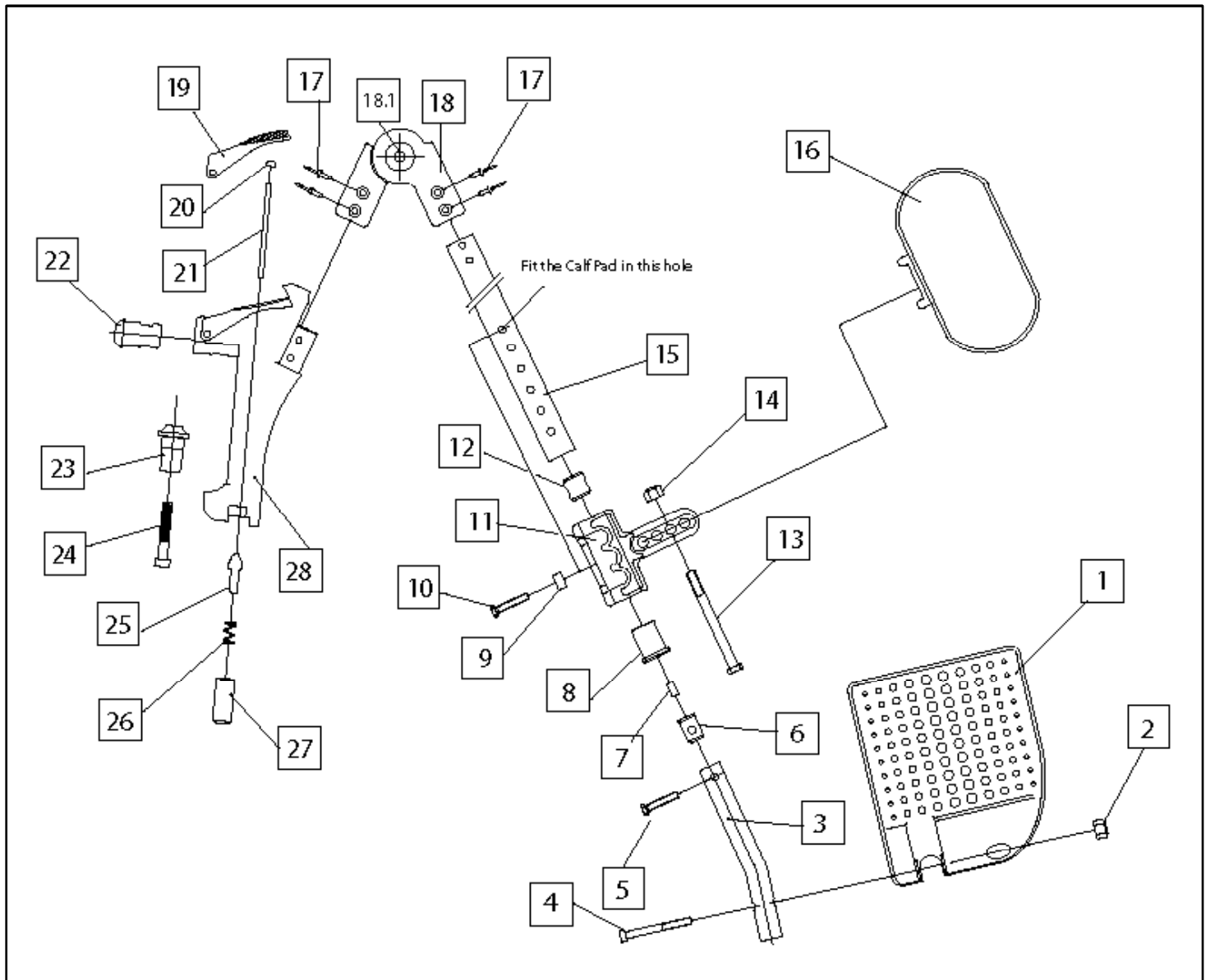


Pos	Item number	Description	Comment
1-28	21B/I/36	Ass. complete 37/40 cm - LH	
1-28	21B/D/36	Ass. complete 37/40 cm - RH	
1-28	21B/I	Ass. complete 43/46/49/52 cm - LH	
1-28	21B/D	Ass. complete 43/46/49/52 cm - RH	
1	P01307D	Right footplate width 37/40 cm	
1	P01307I	Left footplate width 37/40 cm	
1	P01308D	Right footplate width 43/46/49/52 cm	
1	P01308I	Left footplate width 43/46/49/52 cm	
2	P04013	Din 985 M.6 nut	
3	P11063N	Extension tube	

Breezy 300 - Elevating legrests up until 07/05/2012 (no. 30012050005)

Pos	Item number	Description	Comment
4	P04197	Din 912 M.6 x 60 bolt	
5	P04130	Din 933 M.6 x 25 bolt	
6	P01156	Fastening stud	
7	P04230	Din 557 M.6 nut	
8	P01189	Final end	
9	P02284	End socket	
10	P04101	Din 933 M.6 x 30 bolt	
11	P01116	Calf pad bracket	
12	P03156	Fastening stud	
13	P04129	Din 931 M.8 x 90 bolt	
14	P04006	Din 985 M.8 nut	
15	P05570	Leg elevating tube	
16	P01228	Calf pad	
17	P04200	4.8x14 rivet	
18	C05196	Link	
18.1	P01277	Release button	
19	P01198	Handle cover	
20	P01310	Exterior plug	
21	P11192	Pushbar	
22	P01190	Upper plug	From 01/09/05
23	P01182	Upper guide	From 01/09/05
24	P04338	CL 81 Z 6 x 45 bolt	From 01/09/05
25	P01191	Unlocking pivot	
26	P15035	Spring	
27	P03139	Upper socket	
28	P11296	Support for elevating footrests	From 01/09/05

Breezy 300 - Elevating legrests from 07/05/2012 (no. 30012050005)

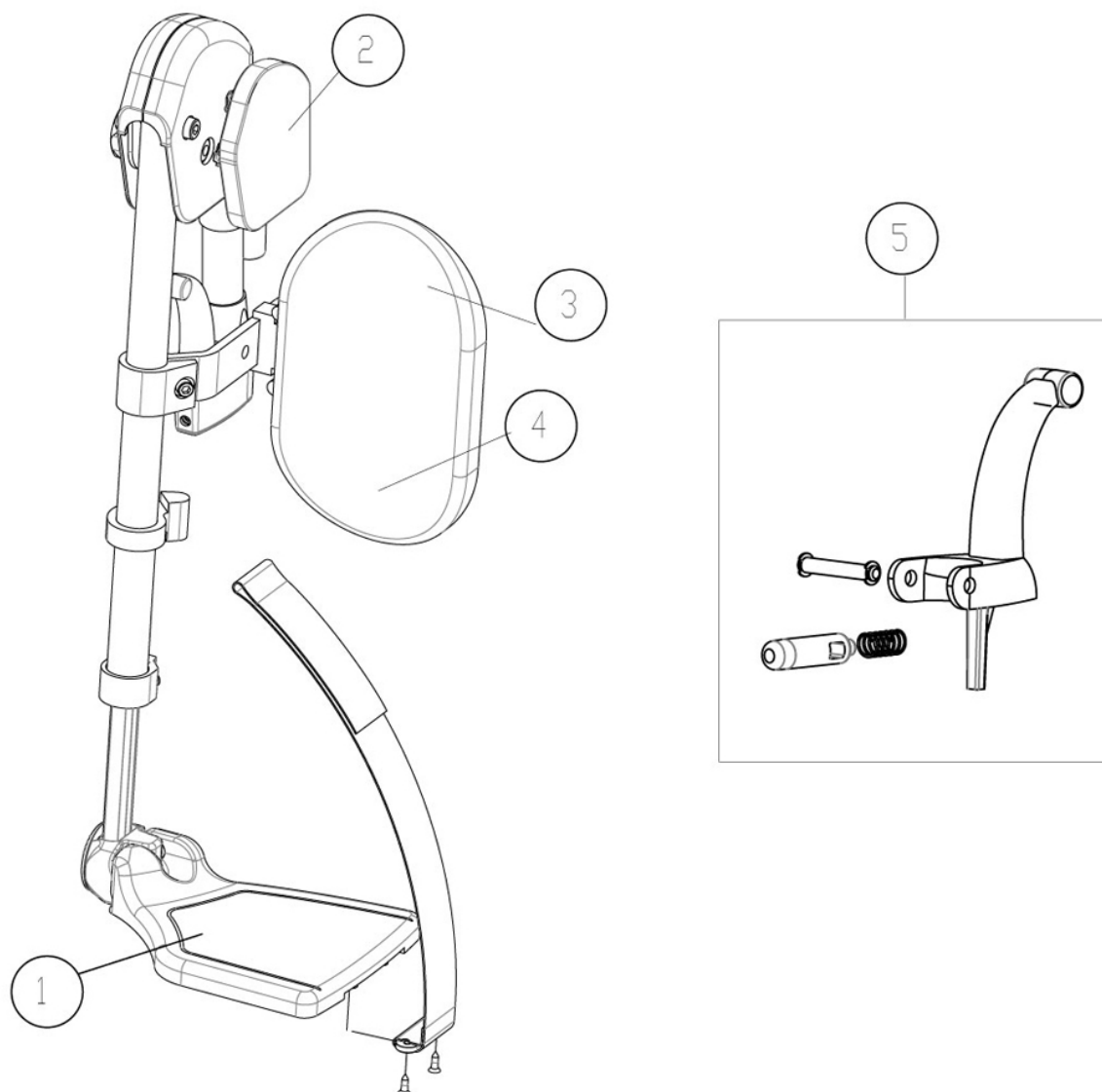


Pos	Item number	Description	Comment
s/n	064260-000	Elevating legrest assembly (without platform)	Pair
s/n	064260-002	Left elevating legrest assembly (without platform)	LH
s/n	064260-001	Right elevating legrest assembly (without platform)	RH
1	064200-101	Composite footboard 157 mm (with heel band and extension tube) SW 37-40 cm	Pair
1	064200-111	Composite footboard 157 mm (with heel band and extension tube) SW 37-40 cm	LH
1	064200-121	Composite footboard 157 mm (with heel band and extension tube) SW 37-40 cm	RH
1	064200-102	Composite footboard 182 mm (with heel band and extension tube) SW 43/46/49/52 cm	Pair
1	064200-112	Composite footboard 182 mm (with heel band and extension tube) SW 43/46/49/52 cm	LH

Breezy 300 - Elevating legrests from 07/05/2012 (no. 30012050005)


Pos	Item number	Description	Comment
1	064200-122	Composite footboard 182 mm (with heel band and extension tube) SW 43/46/49/52 cm	RH
2	064250-001	Knee pad	RH
2	064250-002	Knee pad	LH
3 - 4	064924	Calf pad with cover	
4	064927	Calf pad cover	
5	064999-ET	Unlocking handle cover complete	Pack of 10

Breezy 300 - Fixed footrests

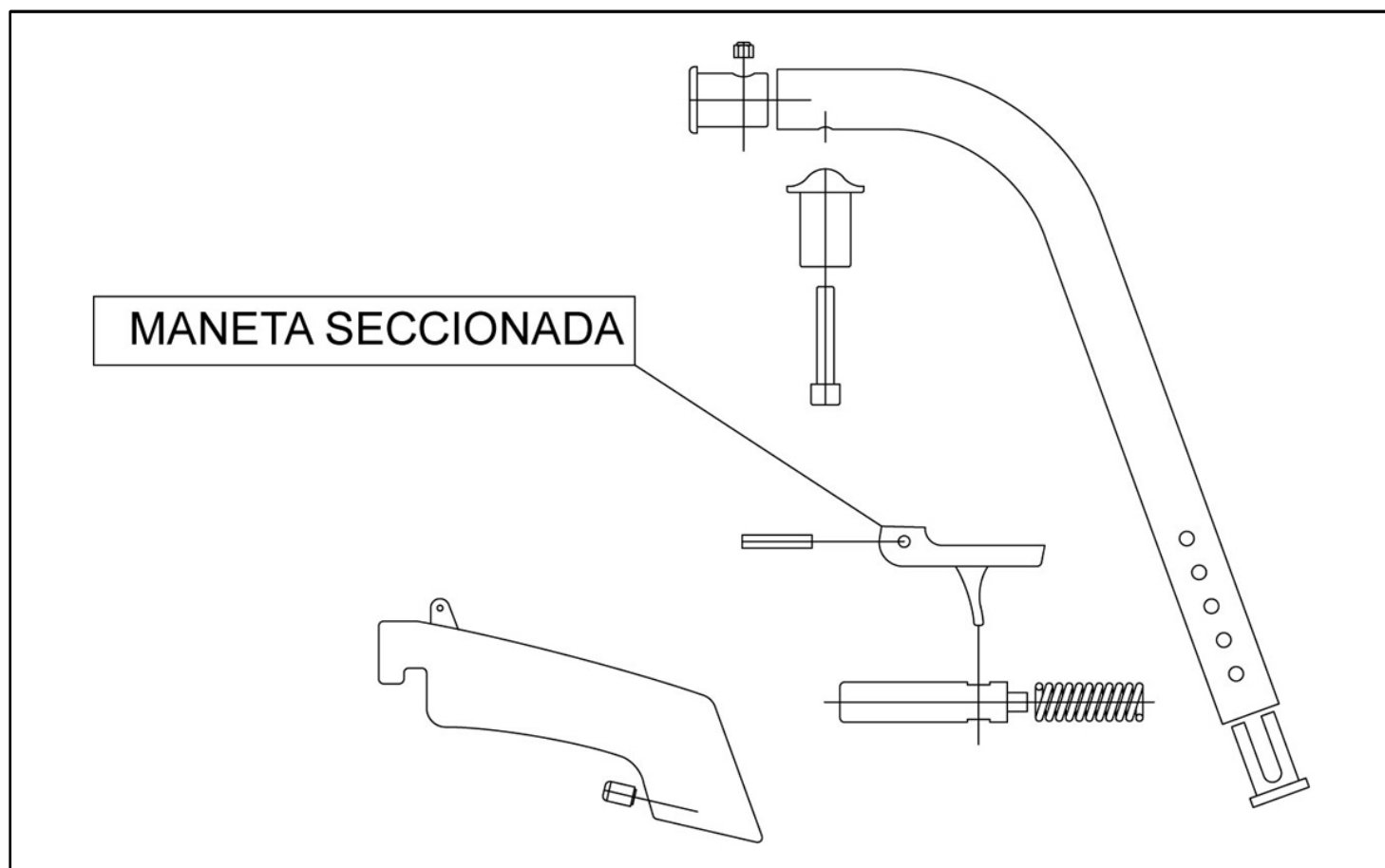

Pos	Item number	Description
s/n	C05347GS	Fixed footrests, pair SELENIUM GREY, complete assembly
s/n	C05347RB	Fixed footrests, pair BRILLIANT RED, complete assembly
s/n	C05347VM	Fixed footrests, pair APPLE GREEN, complete assembly
s/n	C05347NB	Fixed footrests, pair BRILLIANT BLACK, complete assembly


Breezy 300 - Footboard up until 07/05/2012 (no. 30012050005)

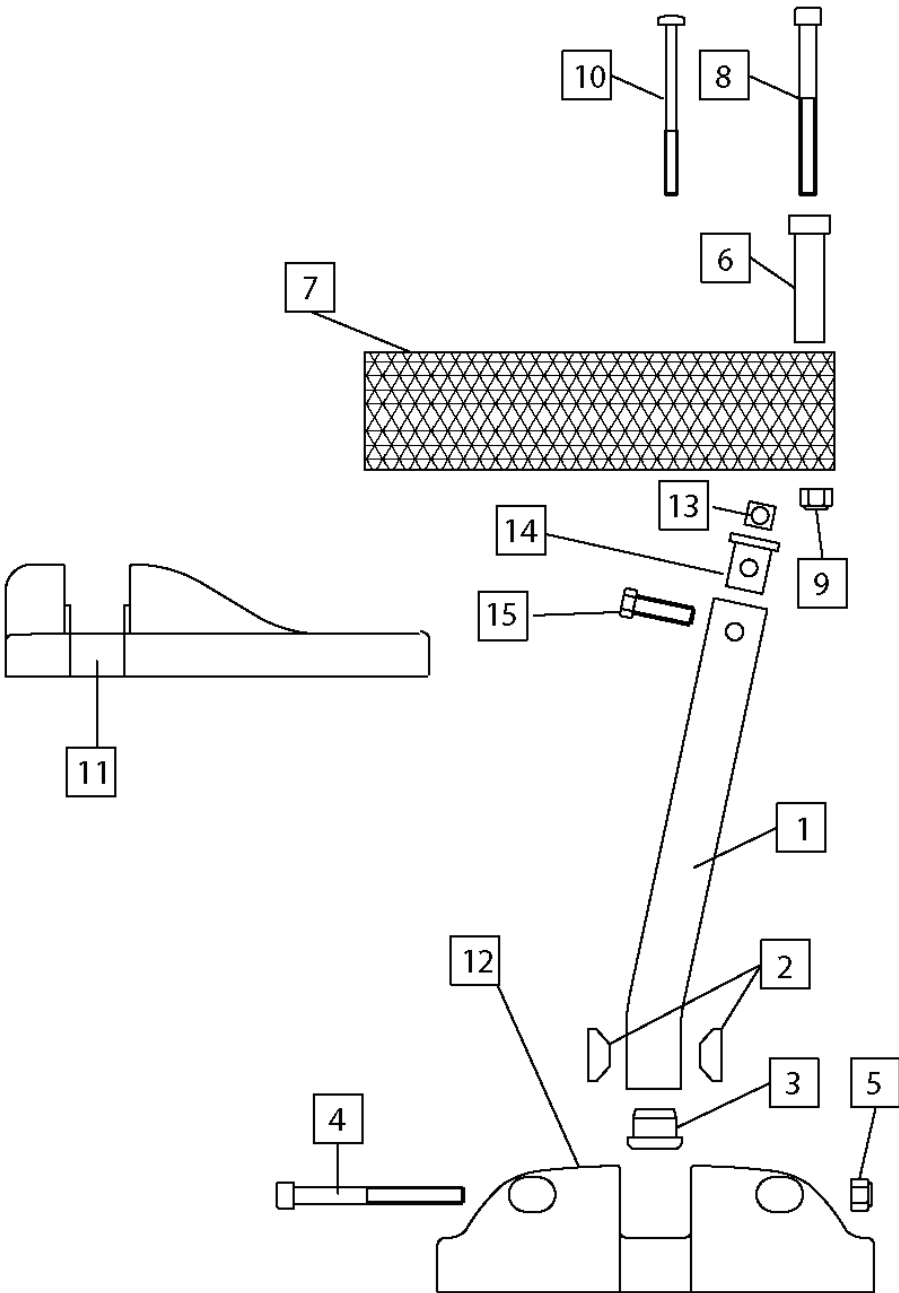

Pos	Item number	Description	Comment
1-10	C05136	Ass. footboard 40, 46, 52 cm - RH	
1-10	C05137	Ass. footboard 40, 46, 52 cm - LH	
1-10	C05138	Ass. footboard 37 cm - RH	


Pos	Item number	Description	Comment
1-10	C05139	Ass. footboard 37 cm - LH	
1	P11063 N	Aluminium tube	
2	P01186	16 half stem washer	
3	P01192	Plug	Up until 1/04/05
4	P04197	Din 912 M.6 x 60 bolt	
5	P04013	Din 985 M.6 nut	
6	P01187	Support	
7	P08169	Heel Loop 40, 46, 52 cm	
8	P01497	Din 912 M.6 x 60 bolt	Up until 1/07/05
9	P04013	Din 985 M.6 nut	Up until 1/07/05
10	P04336	5 x 50 Alda PH bolt	From 1/07/05
11	P01307D	Platform 37 cm Right	
11	P01307I	Platform 37 cm Left	
12	P01204	Platform 40-43-46-52 cm	
13	P04230	Square nut M6 - Din 557	
14	P01156	Footrests fastening stud	
15	P04130	Din 933 M6 x 25 bolt	

Breezy 300 - Footboard from 07/05/2012 (no. 30012050005)

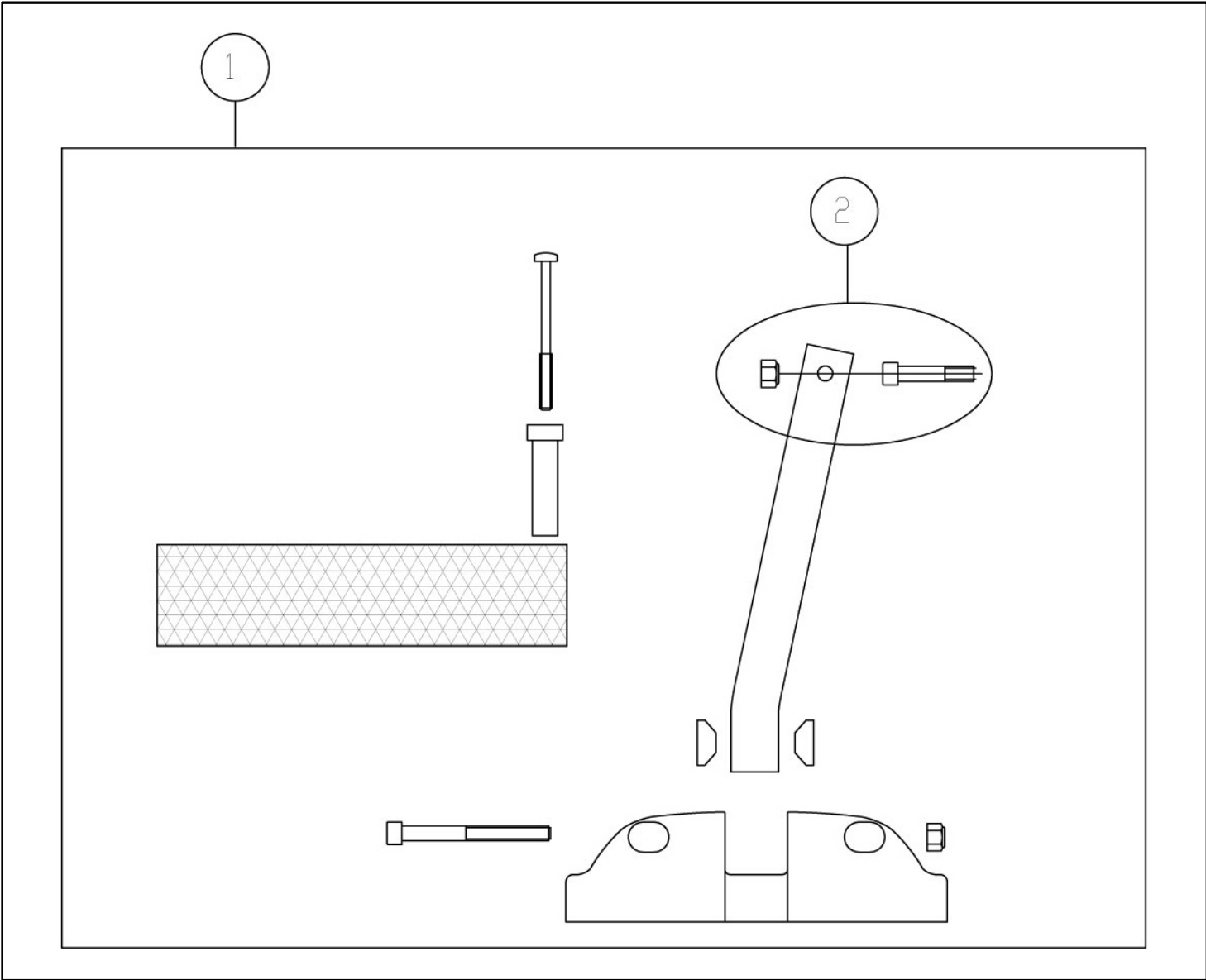


Pos	Item number	Description
1	C05138	Right platform assembly 37 cm
1	C05139	Left platform assembly 37 cm
1	C05136	Right platform assembly 40, 43, 46, 49, 52 cm
1	C05137	Left platform assembly 40, 43, 46, 49, 52 cm
1	C05324	Platform assembly, right 37 cm, SHORT FOOTRESTS
1	C05325	Platform assembly, left 37 cm, SHORT FOOTRESTS
1	C05326	Platform assembly, right 40, 43, 46, 49, 52 cm, SHORT FOOTRESTS
1	C05327	Platform assembly, left 40, 43, 46, 49, 52 cm, SHORT FOOTRESTS
2	C04114	Tie-down kit

Breezy 300 - Continuous footboard up until 07/05/2012 (no. 30012050005)

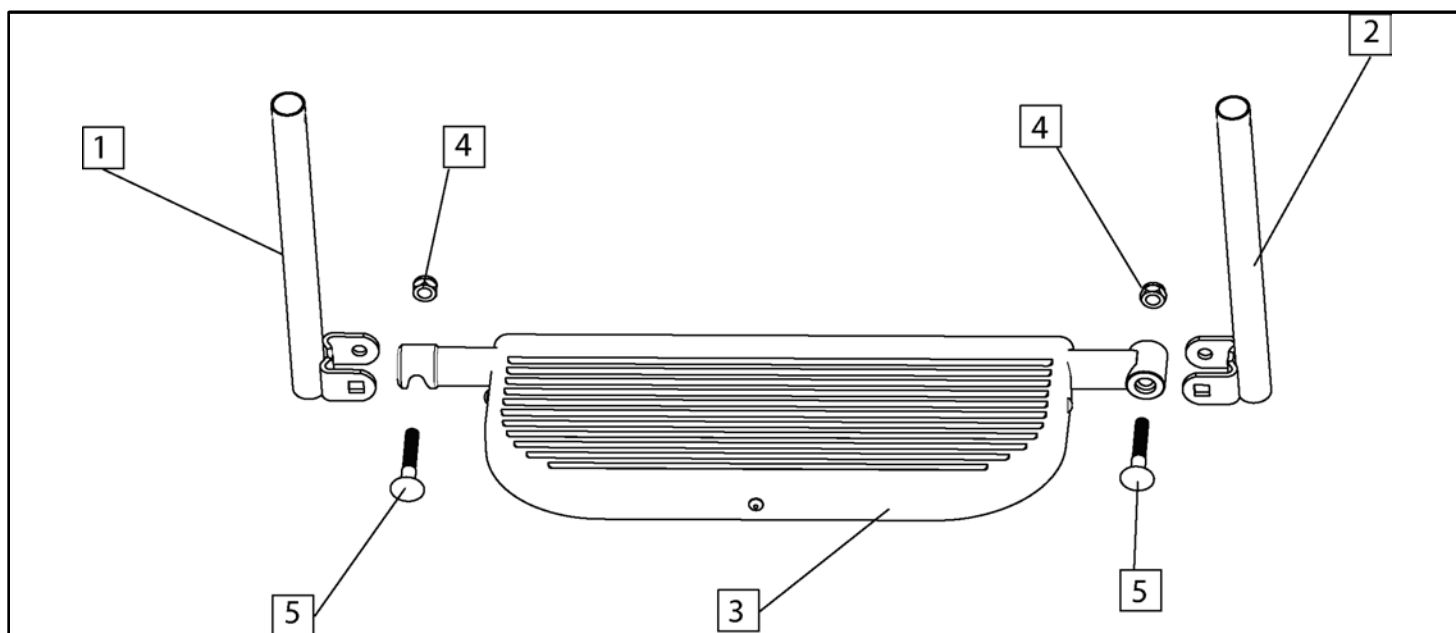


Pos	Item number	Description
1-5	22B/37	Platform assembly 37 cm
1-5	22B/40	Platform assembly 40 cm
1-5	22B/43	Platform assembly 43 cm
1-5	22B/46	Platform assembly 46 cm
1-5	22B/49	Platform assembly 49 cm
1-5	22B/52	Platform assembly 52 cm
1	C05076	Right telescopic support
2	C05077	Left telescopic support
3	C05294	Platform 37 cm
3	C05295	Platform 40 cm
3	C05296	Platform 43 cm
3	C05297	Platform 46 cm
3	C05298	Platform 49 cm
3	C05299	Platform 52 cm
4	P04006	M8 nut
5	P04024	Din 603 M8 x 40 bolt

Breezy 300 - Continuous footplate from 07/05/2012 (no. 30012050005)

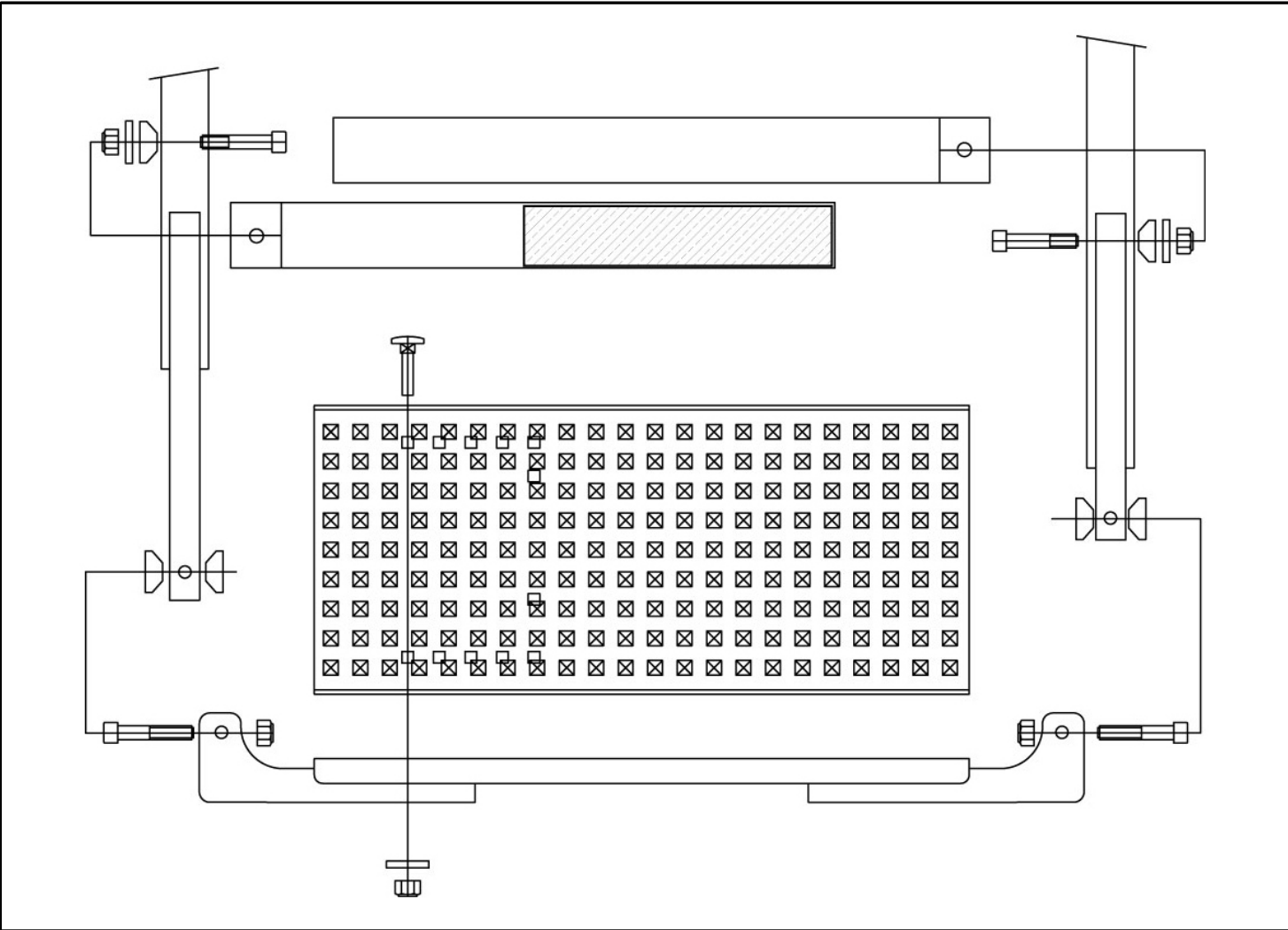


Pos	Item number	Description
s/n	C05334	Continuous footplate 37 cm
s/n	C05335	Continuous footplate 40, 43, 46, 49, 52 cm
s/n	C05336	Continuous footplate for SHORT FOOTRESTS, 40, 43, 46, 49, 52 cm

Breezy 300 - Angle-adjustable footplate

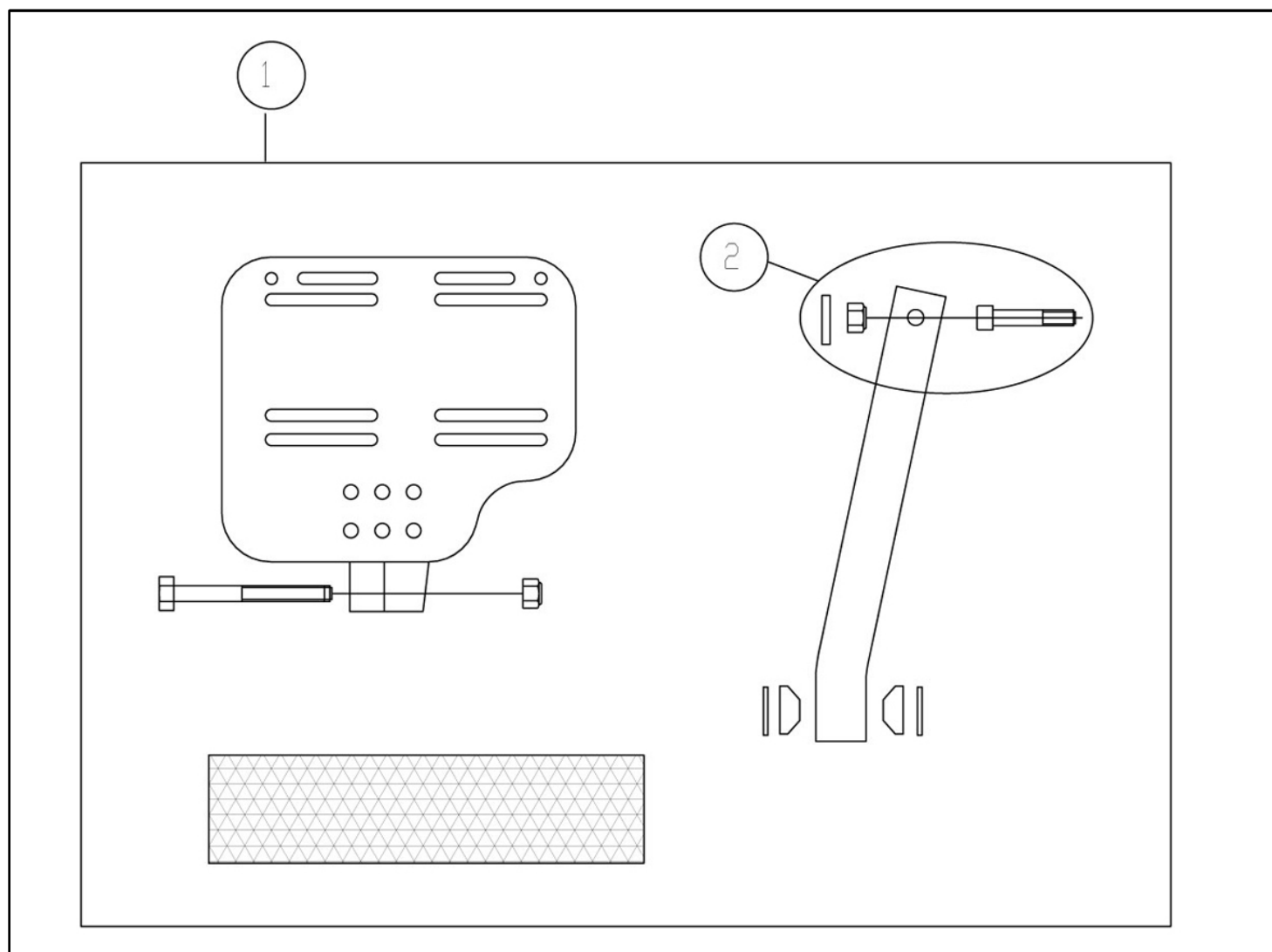


Pos	Item number	Description
1	C05330	Right plastic footplate, angle adjustable SW, 40, 43, 46, 49, 52 cm
1	C05331	Left plastic footplate, angle adjustable SW; 40, 43, 46, 49, 52 cm
1	C05332	Right plastic footplate, angle adjustable SW, 40, 43, 46, 49, 52 cm SHORT FOOTRESTS
1	C05333	Left plastic footplate, angle adjustable SW 40, 43, 46, 49, 52 cm SHORT FOOTRESTS
2	C04114	Tie-down kit

Breezy 300 - 6" solid wheel

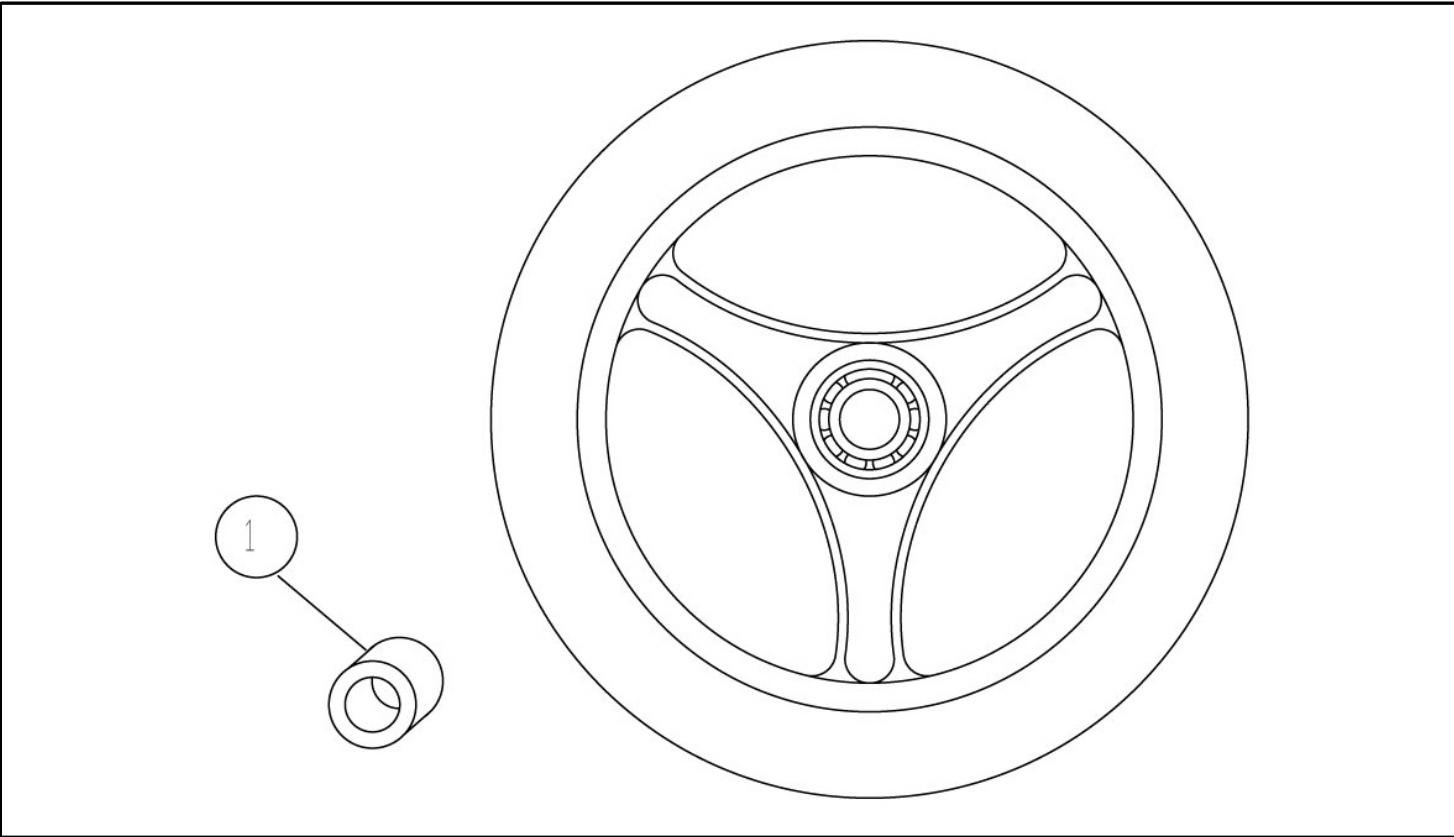


Pos	Item number	Description
s/n	C06124	6" solid wheel assembly
1	P11166	Socket

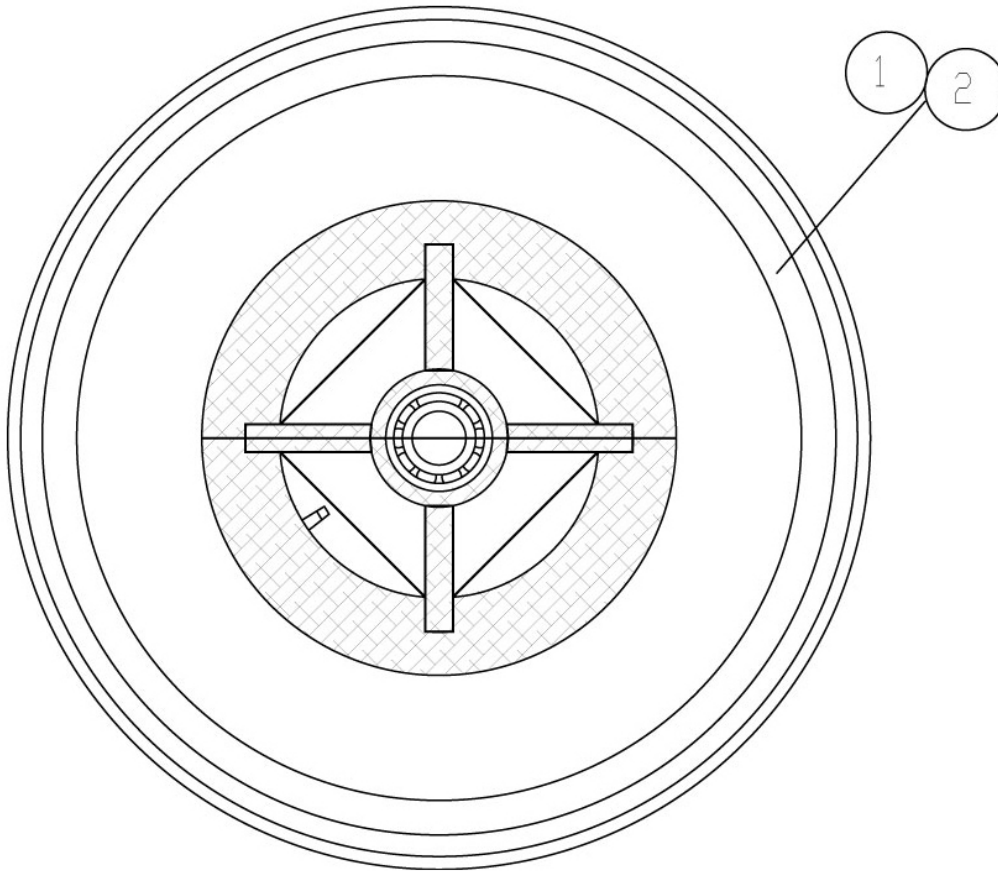Breezy 300 - 8" x 2" pneumatic tyre


Pos	Item number	Description
s/c	C06005	8x2" pneumatic tyre assembly
1	P09023	8" grey tyre
2	P09024	8" grey inner tube

Breezy 300 - 8" x 1" solid wheel

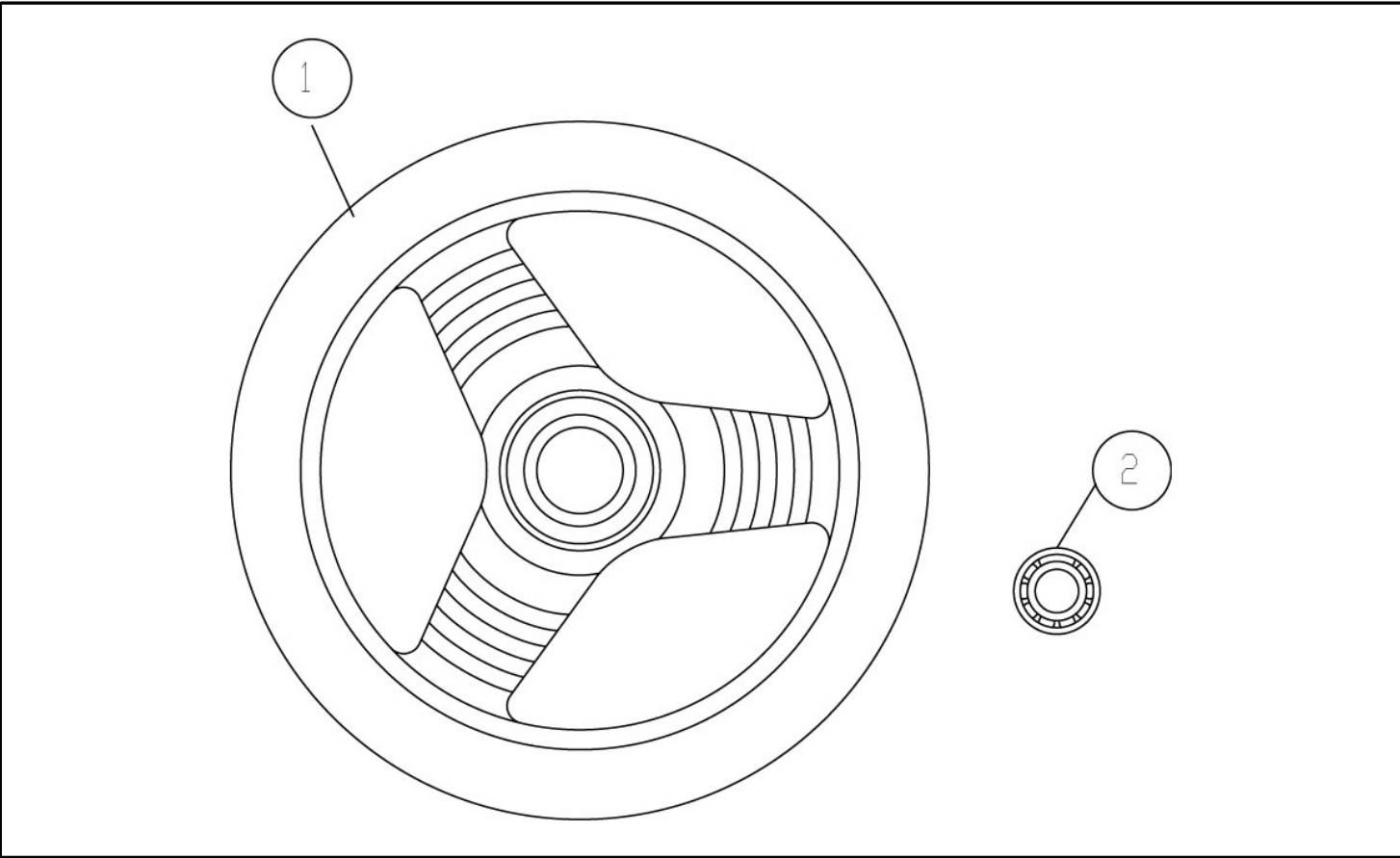


Pos	Item number	Description
s/n	C06057	8" x 1" solid wheel assembly
1	C06056	Assembly without bearings
2	P10012	23 x 8 x 30 bearing

Breezy 300 - 8" x 2" solid wheel

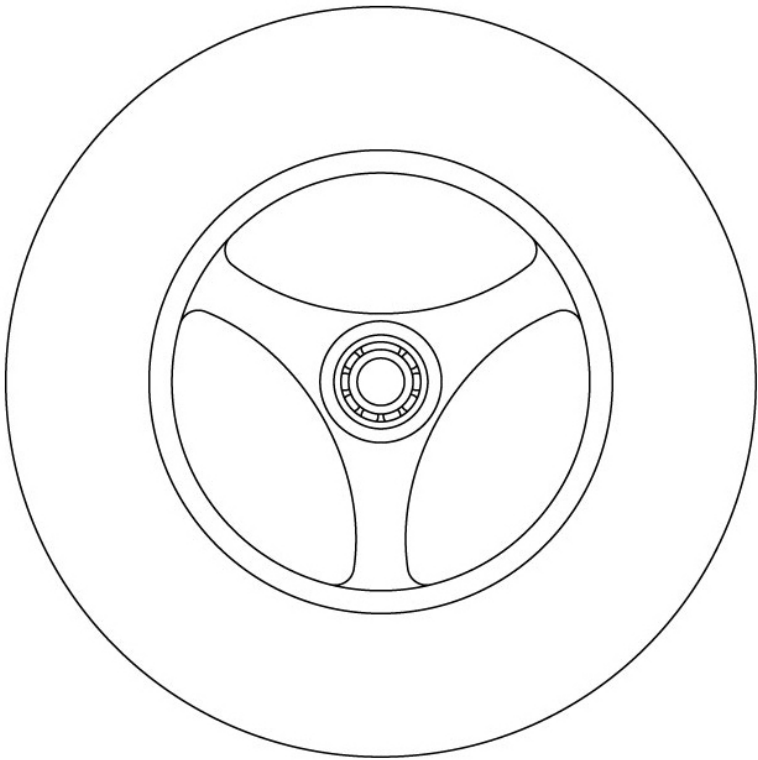


Pos	Item number	Description
1	C06141	8" x 2" solid wheel assembly

Breezy 300 - Fork

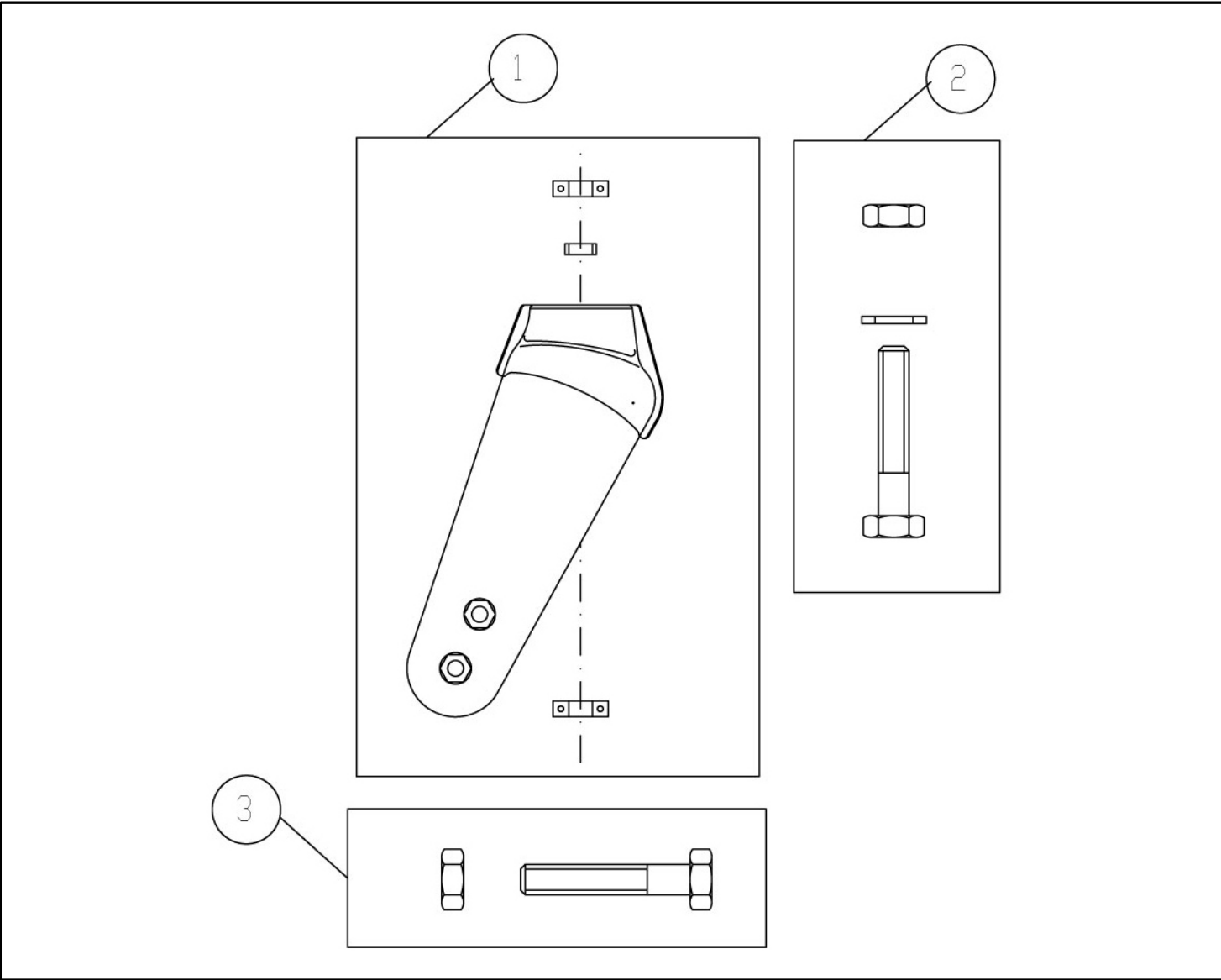


Pos	Item number	Description
1	C06163	Fork assembly with bearings
2	C04074	Fork kit for vertical post
3	C04075	Castor to fork kit Tie

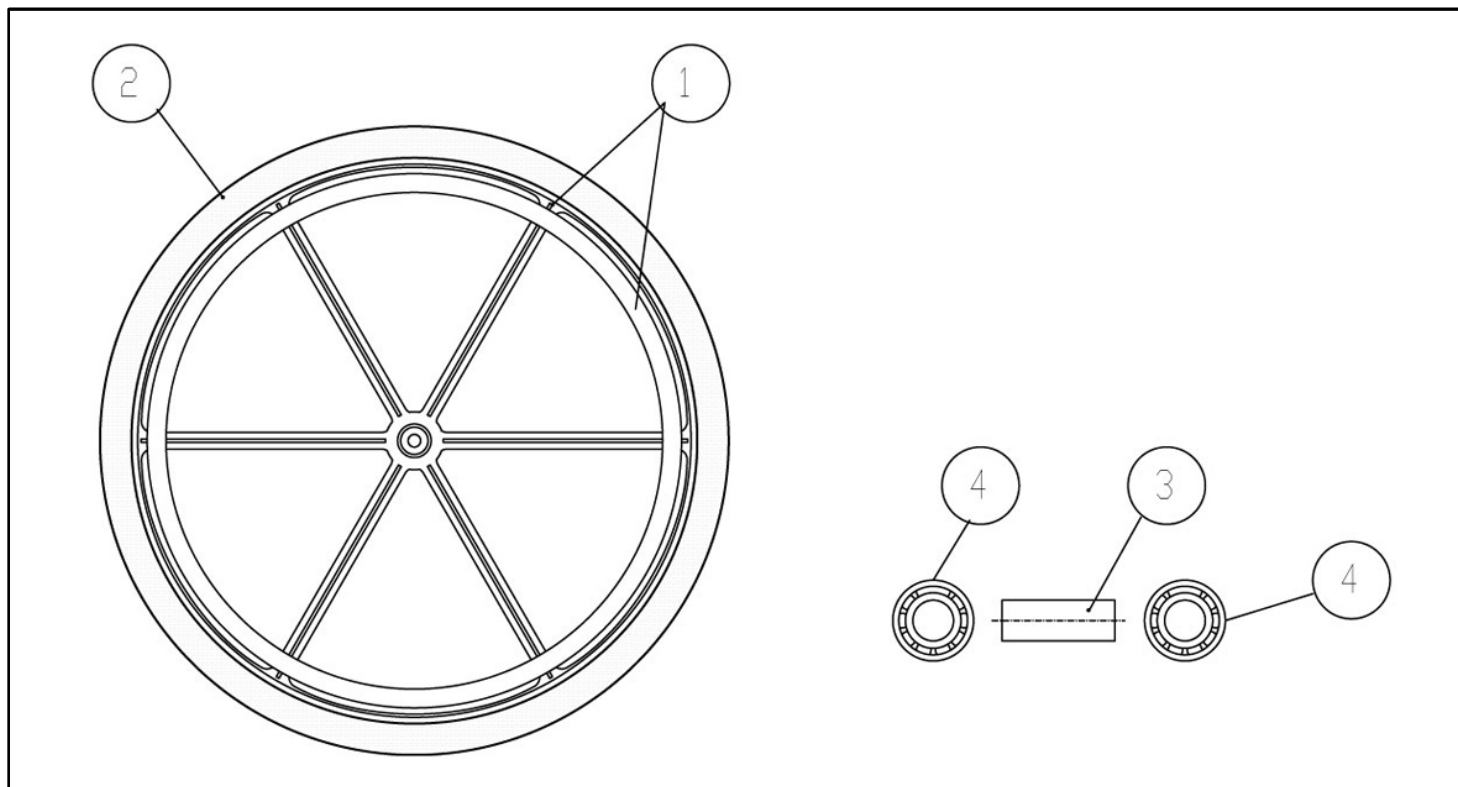Breezy 300 - Sport 24" solid wheel


Pos	Item number	Description
s/n	C06219	Sport 24" solid wheel assembly
1	C06753	Rim with handrim
2	P09070	24" solid wheel
3	P03083	Socket
4	P10011	6001zz 28x12x8 bearing

Breezy 300 - Sport 24" pneumatic tyre

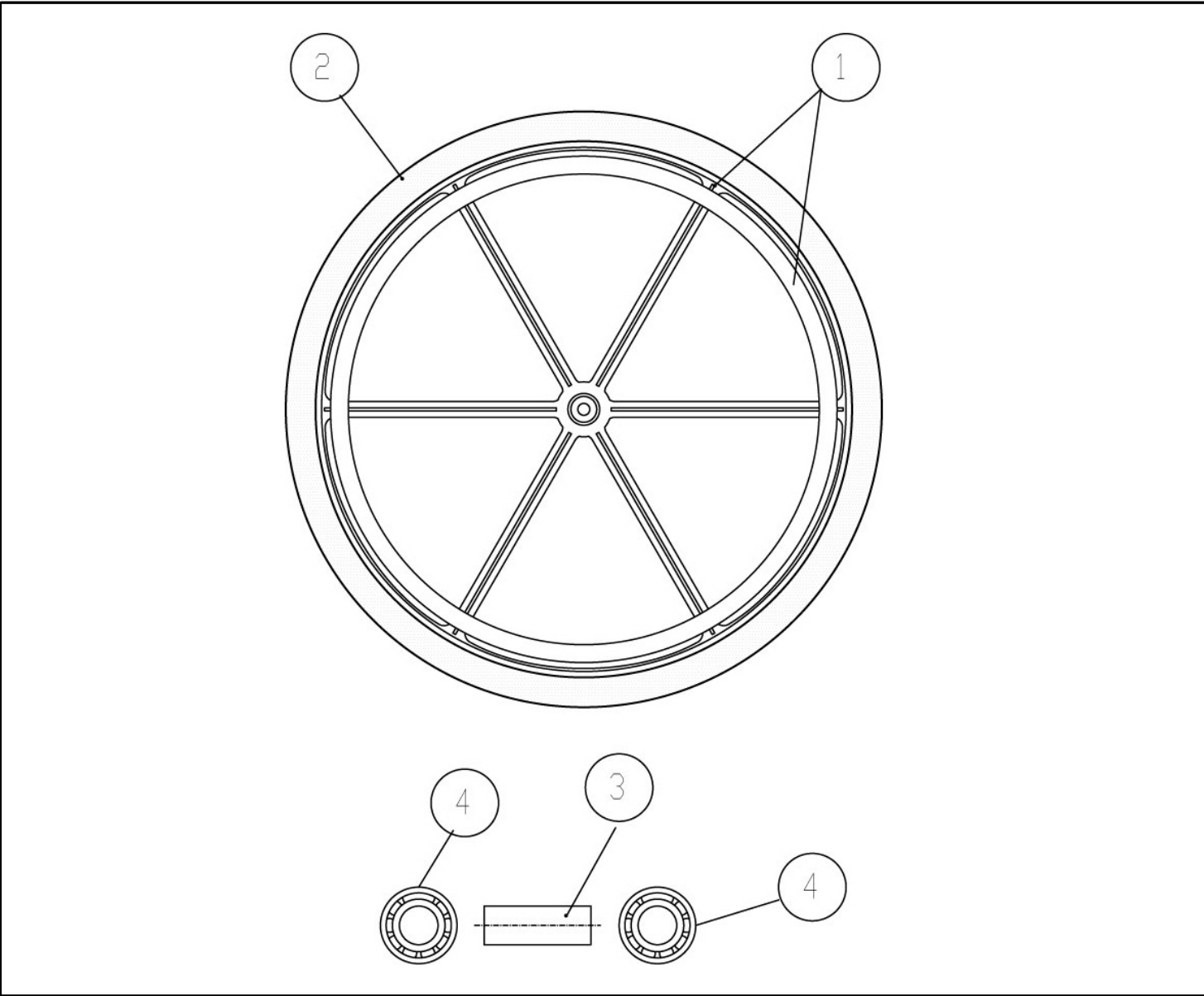


Pos	Item number	Description
s/n	C06023	Sport 24" pneumatic tyre assembly
1	C06753	Rim with handrim
2	P09004	24" inner tube
2	P09005	24" tyre
3	P03083	Socket
4	P10011	6001zz 28x12x8 bearing

Breezy 300 - 24" solid spoke wheel

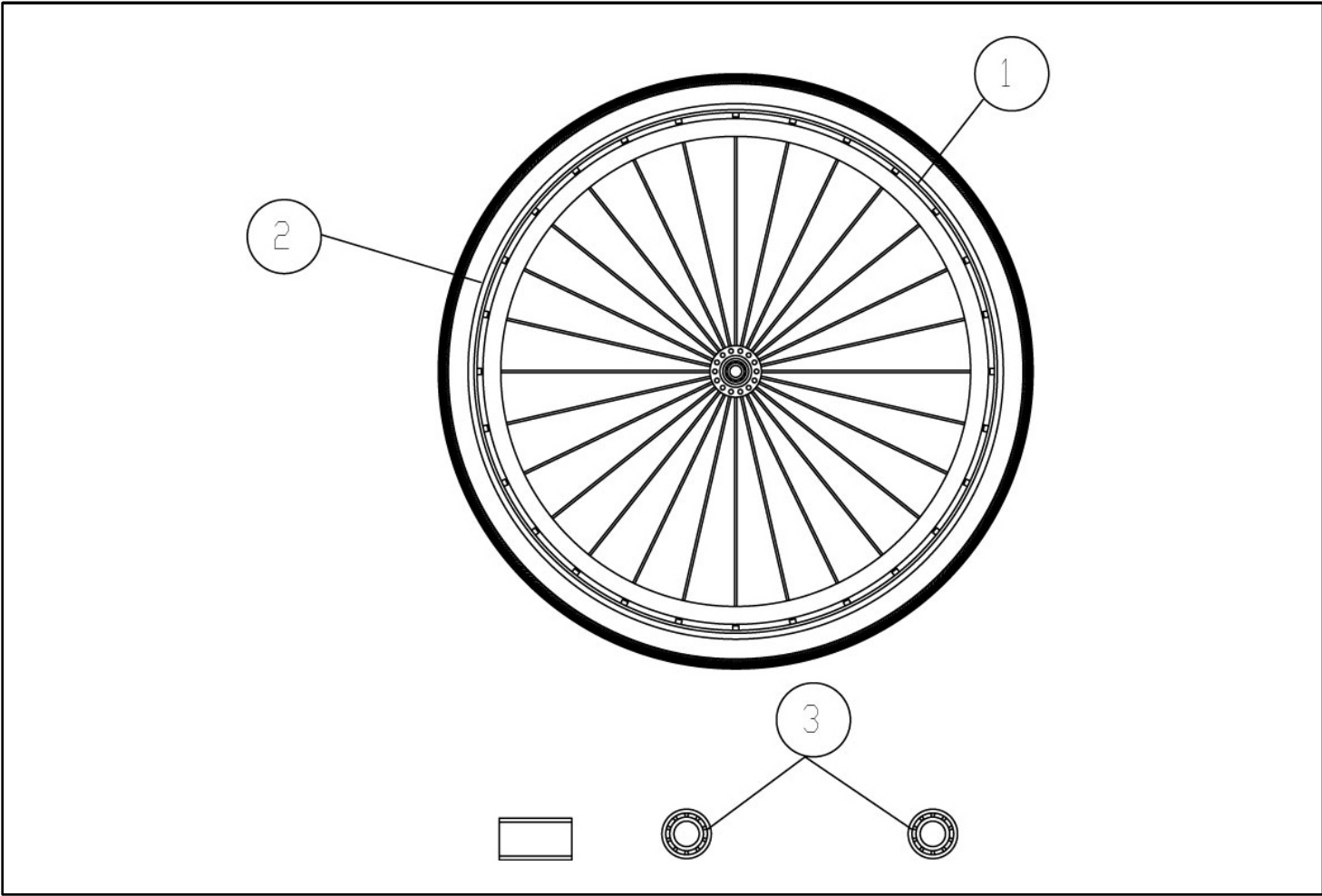


Pos	Item number	Description
1	C06253	24" solid spoke wheel assembly
2	P09082	Solid
3	P10011	6001zz bearing

Breezy 300 - 24" pneumatic wheel with spokes

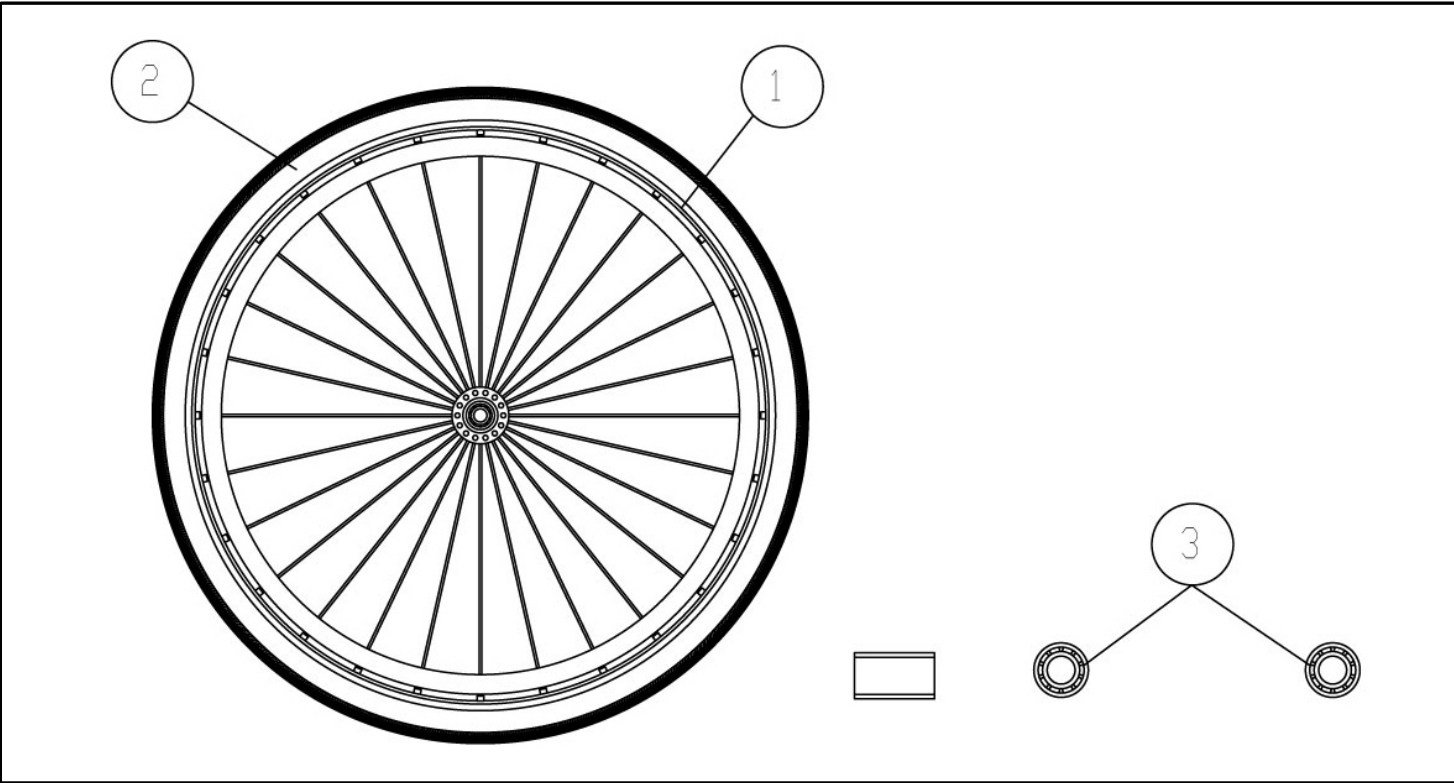


Pos	Item number	Description
1	C06252	24" pneumatic wheel assembly with spokes
2	P09004	24" inner tuve
2	P09005	24" tyre
3	P10011	6001zz bearing


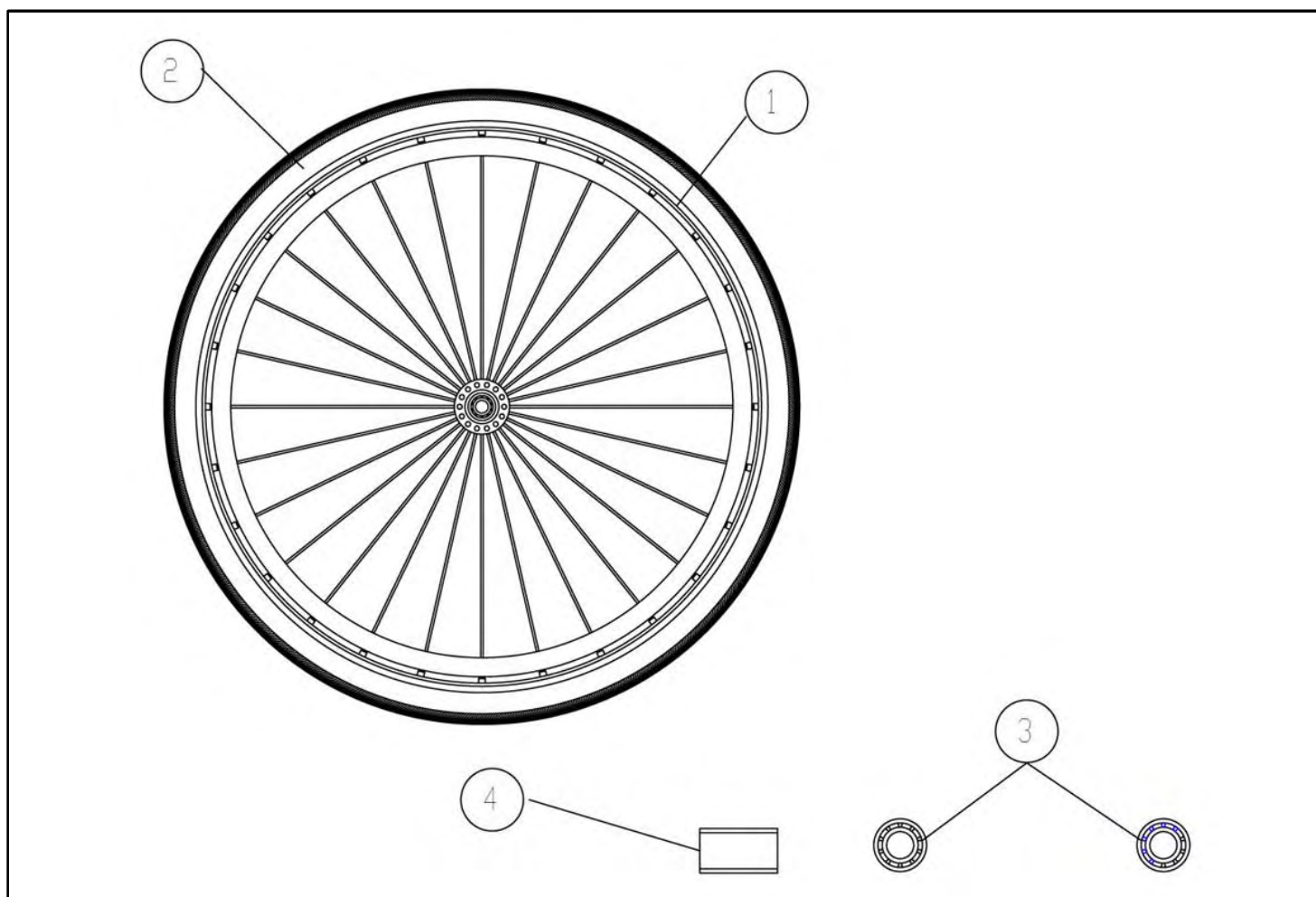
Breezy 300 - 22" pneumatic wheel with spokes

Pos	Item number	Description
s/n	C06122	22" pneumatic wheel assembly with spokes
1	C06131	Rim with handrim
2	P09057	Inner tube
2	P09056	Cap
3	P10011	6001zz bearing
4	P03140	Spacer socket

Breezy 300 - Quick Release Axle

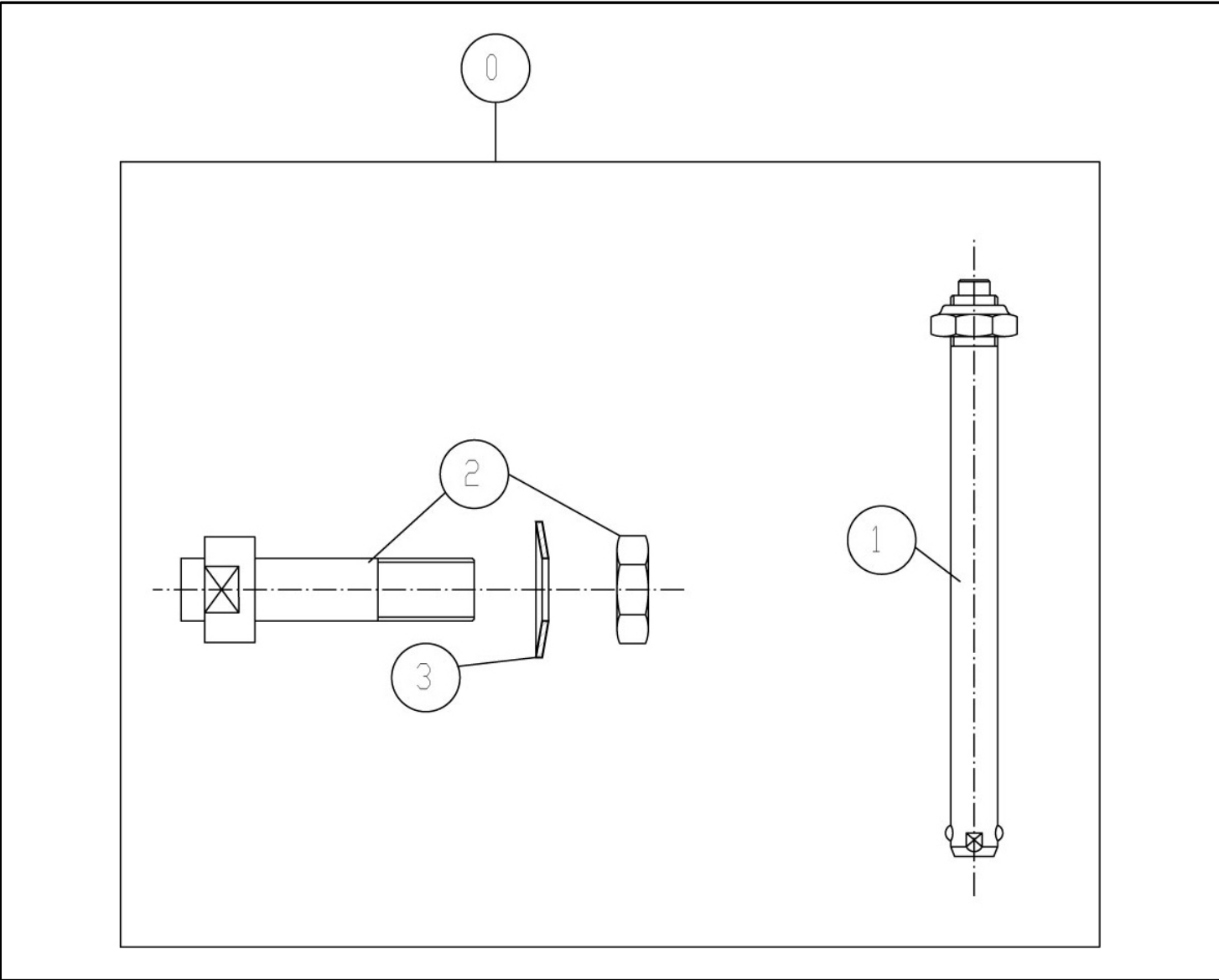


Pos	Item number	Description
0	C011021	Complete quick release axle assembly
1	C06148	Quick Release Axle
2	C06149	Support socket and nut
3	P04302	DIN 127 Grover washer dia. 16

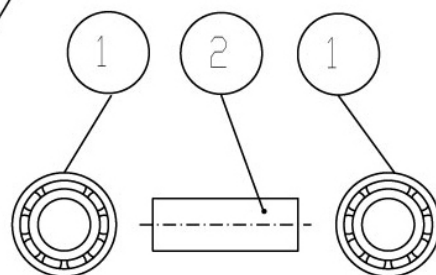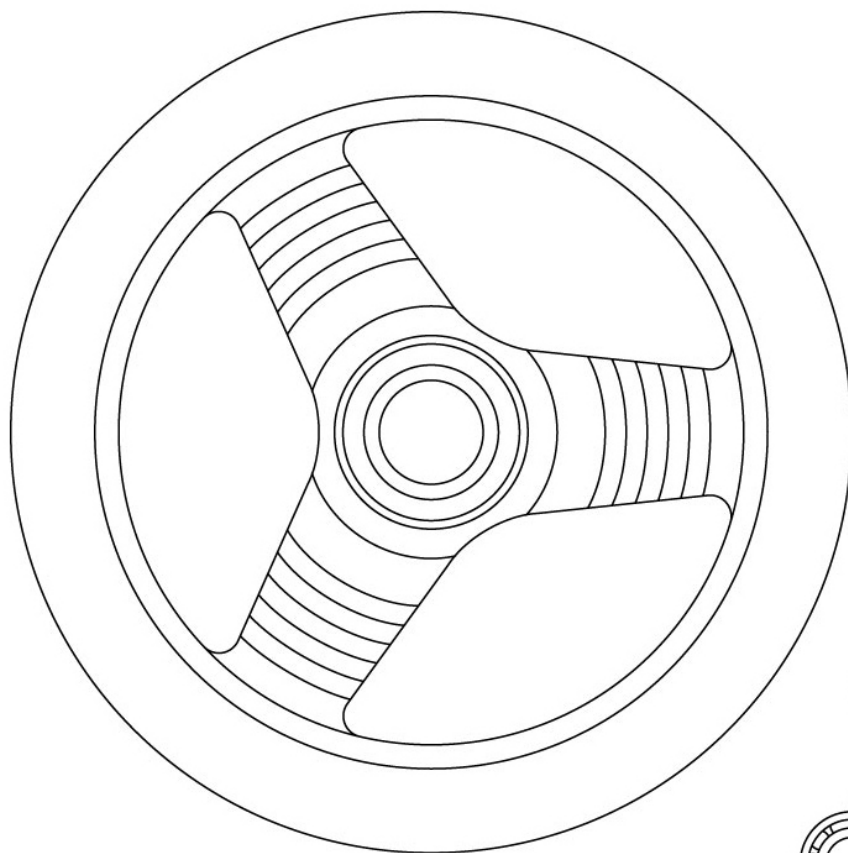Breezy 300 - 12" solid wheel

Pos	Item number	Description
s/n	C06074	12" solid wheel assembly
1	P10005	6201 zz 32x12x10 bearing
2	P03131	Spacer socket


Breezy 300 - 12" pneumatic tyre

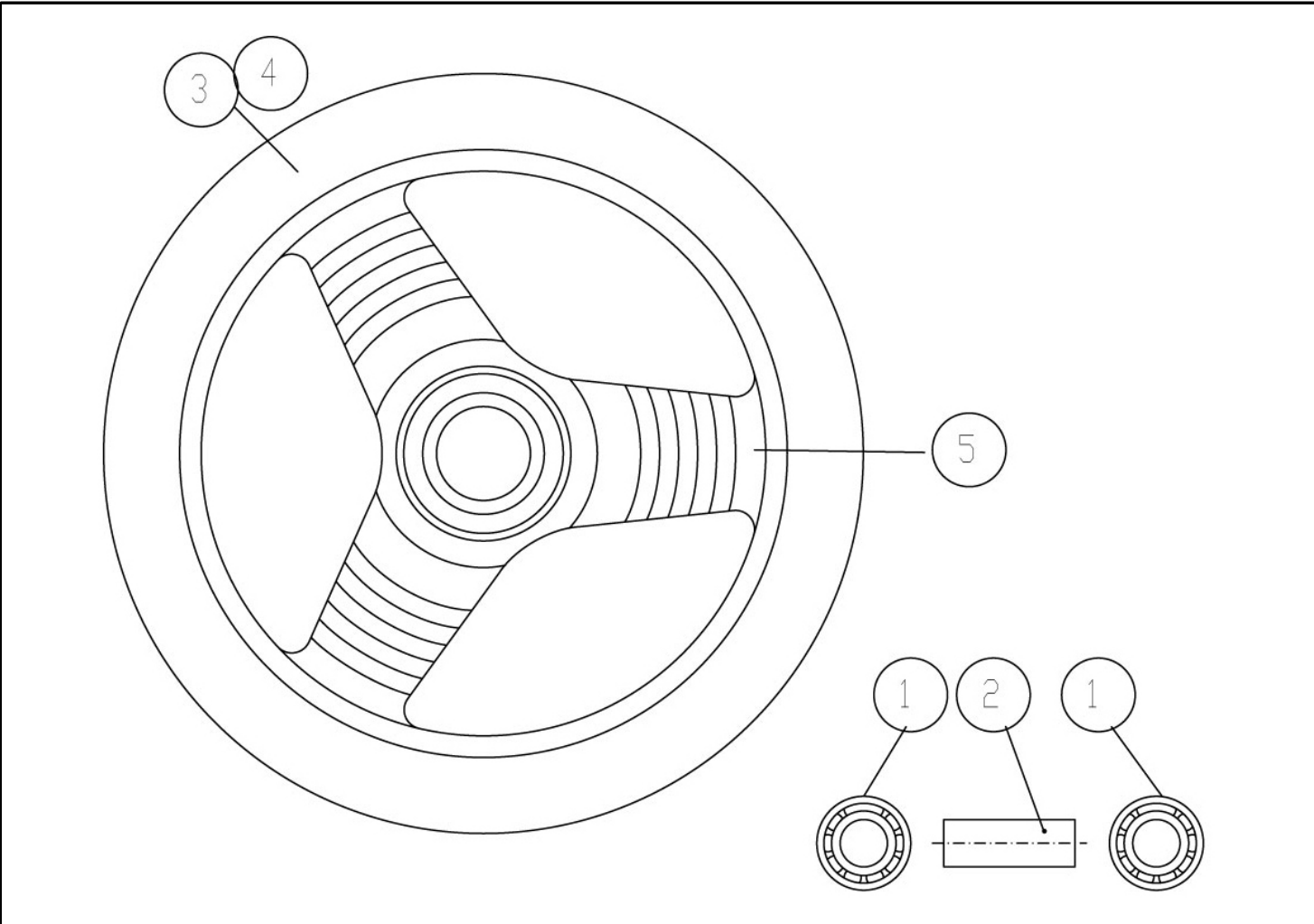

Pos	Item number	Description
s/n	C06075	12" pneumatic tyre assembly
1	P10005	6201 zz 32x12x10 bearing
2	P03131	Spacer socket
3	P09007	12" inner tube
4	P09008	12" tyre
5	C06087	Rim


Breezy 300 - Hemiplegic Wheel

Note: When placing your order, please specify whether you need the part for right or left hand side

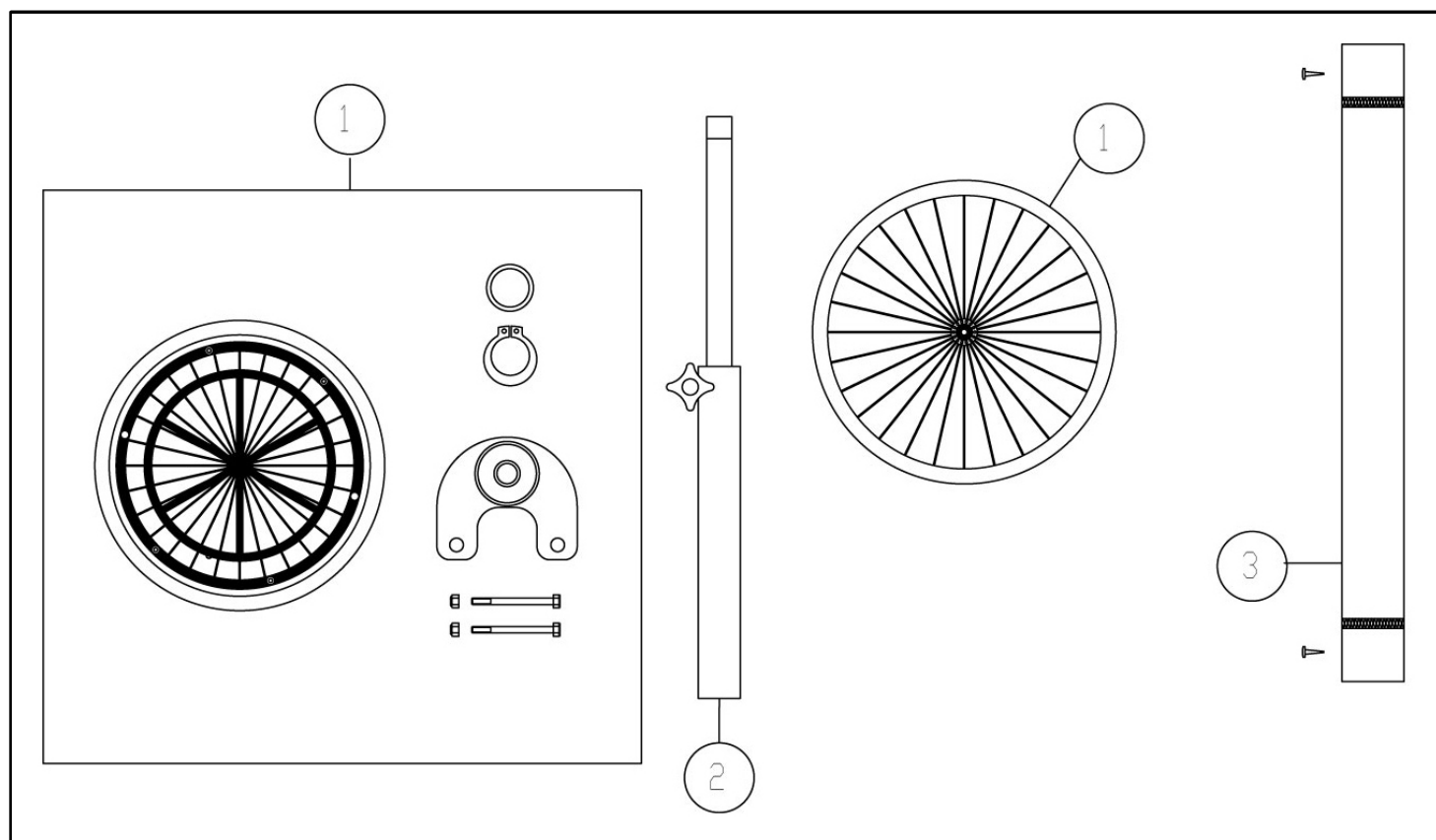


Pos	Item number	Description
1	C011025	One arm drive assembly 37 cm
1	C011026	One arm drive assembly 40 cm
1	C011027	One arm drive assembly 43 cm
1	C011028	One arm drive assembly 46 cm
1	C011029	One arm drive assembly 49 cm
1	C011030	One arm drive assembly 52 cm
2	C06228	Extendible axle for chairs 37, 40, 43, 46 cm
2	C06232	Extendible axle for chairs 49 y 52 cm
3	P08223	Rear wheel anti-opening strap 37 cm
3	P08224	Rear wheel anti-opening strap 40 cm
3	P08225	Rear wheel anti-opening strap 43 cm
3	P08226	Rear wheel anti-opening strap 46 cm
3	P08227	Rear wheel anti-opening strap 49 cm
3	P08228	Rear wheel anti-opening strap 52 cm

Breezy 300 - Brakes up until 07/05/2012 (no. 30012050005)

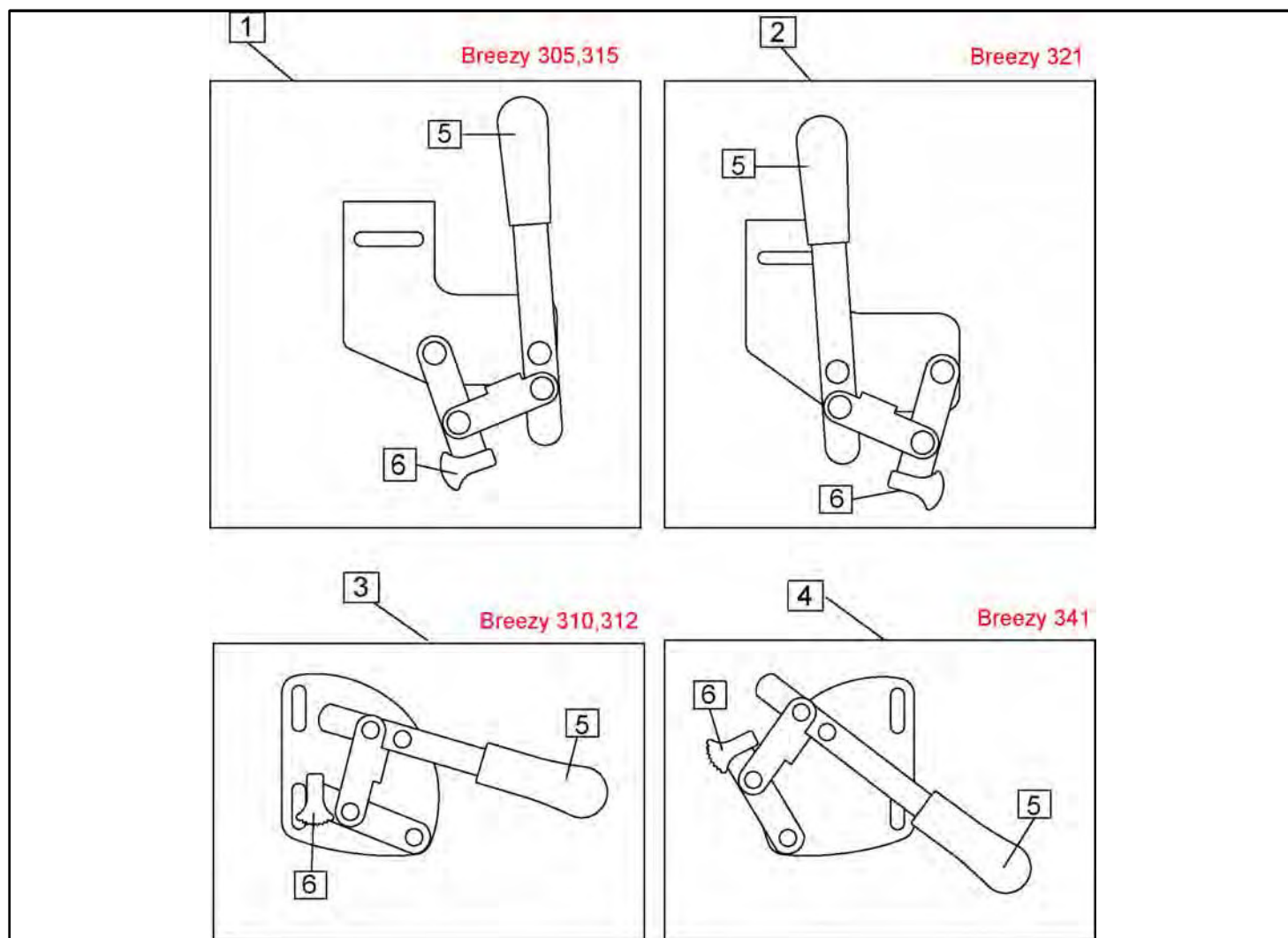


Pos	Item number	Description
1	C03058	Breezy 305, 315 Ass. brake - RH
1	C03059	Breezy 305, 315 Ass. brake - LH
2	C03056	Breezy 321 Ass. brake - RH
2	C03057	Breezy 321 Ass. brake - LH
3	C03030	Breezy 310, 312 Ass. brake - RH
3	C03031	Breezy 310, 312 Ass. brake - LH
4	C03034	Breezy 341 Ass. brake - RH
4	C03035	Breezy 341 Ass. brake - LH
5	P01286	Wheel Lock Lever Cover
6	P01273	Brake Shoe Grey

Breezy 300 - Brakes from 07/05/2012 (no. 30012050005)

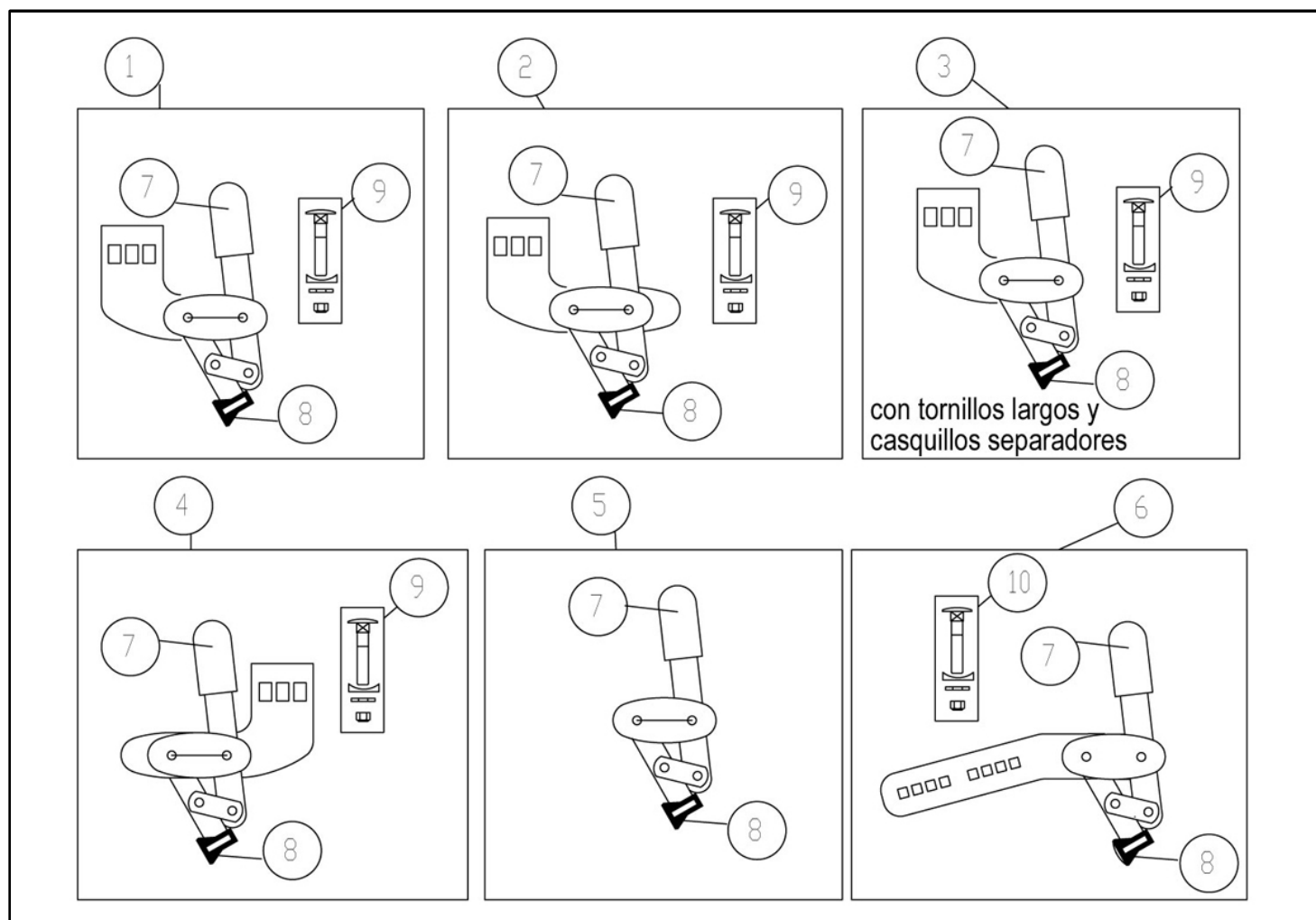


Pos	Item number	Description
1	C03066	Right brake assembly for 24" wheel
1	C03067	Left brake assembly for 24" wheel
2	C03080	Right brake assembly for one arm drive
2	C03081	Left brake assembly for one arm drive
3	C03078	Right brake assembly Quick Release (solid spoke wheel) and 24" drum brake
3	C03079	Left brake assembly Quick Release (solid spoke wheel) and 24" drum brake
4	C03074	Right brake assembly for reclining chair, 24" wheel, and for double amputee
4	C03075	Left brake assembly for reclining chair, 24" wheel, and for double amputee
5	C03072	Right brake assembly for reclining chair, 12" wheel
5	C03073	Left brake assembly for reclining chair, 12" wheel
6	C03070	Right brake assembly for 12" wheel
6	C03071	Left brake assembly for 12" wheel
7	P01286	Brake lever
8	P01273	Brake shoe

Pos	Item number	Description
9	C04079	Brake kit tie
10	C04080	Brake kit tie

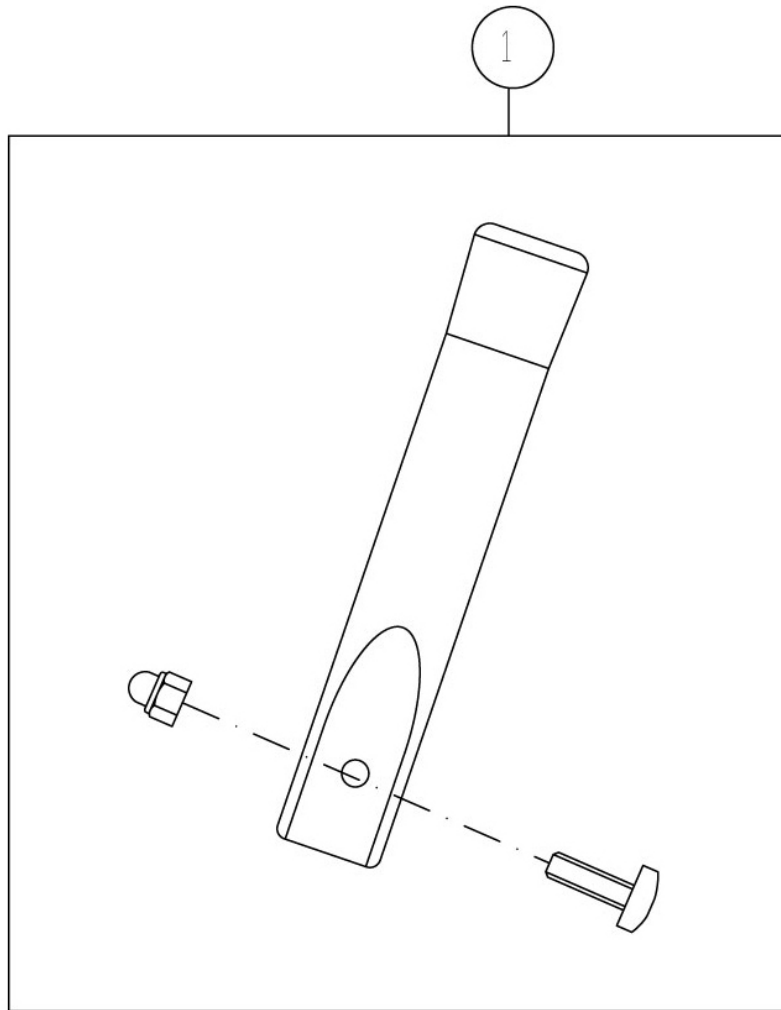Breezy 300 - Brake Extension

Pos	Item number	Description
1	C04087	Brake lever extension assembly


Breezy 300 - Drum brakes for 12" wheel up until 07/05/2012 (no. 30012050005)

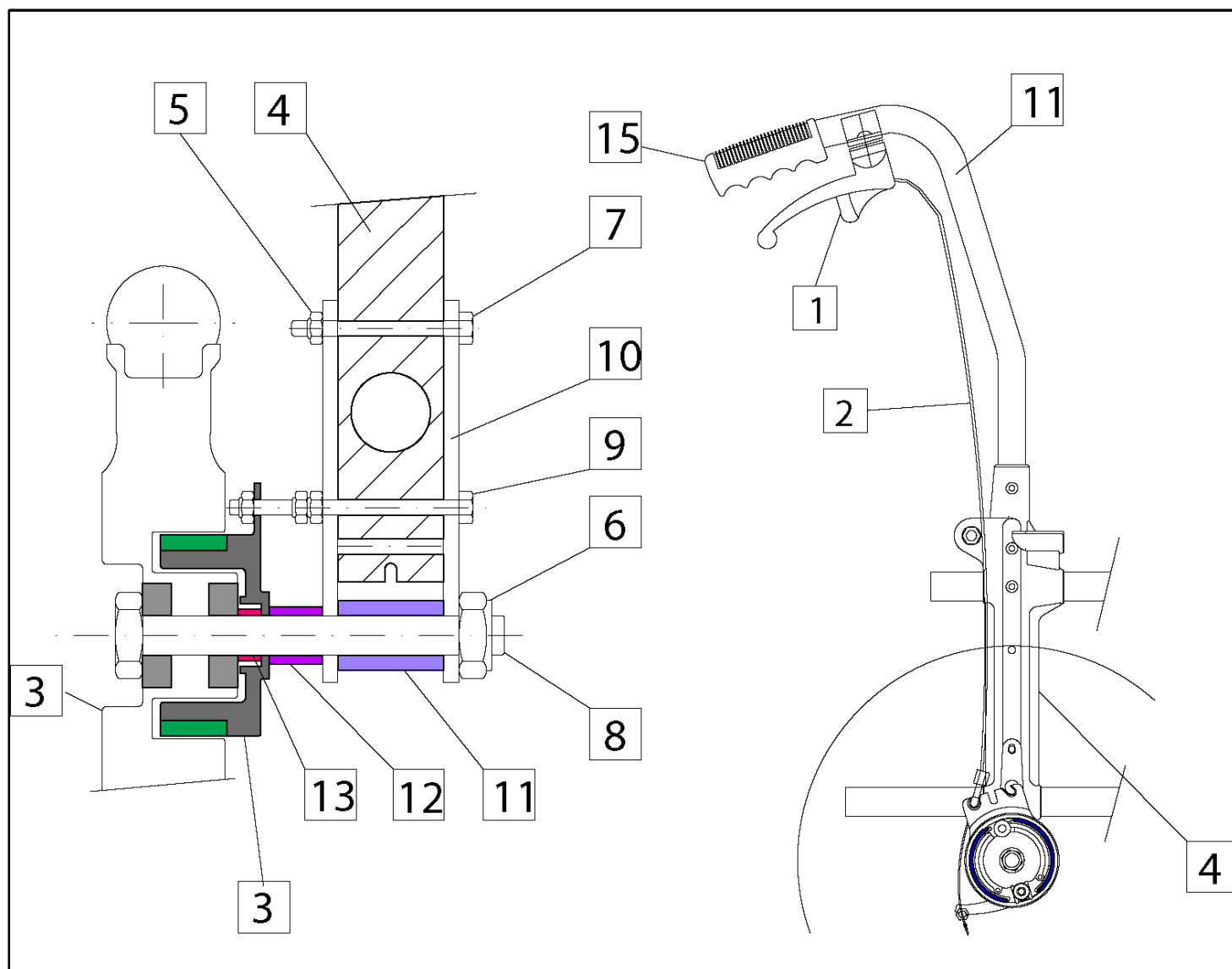

Pos	Item number	Description
	96B/I	Drumbrake assembly - LH
	96B/D	Drumbrake assembly - RH
1	C03018	Brake lever
2	C03020	Cable assembly 0300
3	C06157	Hub + Ø300 wheel right hand
3	C06158	Hub + Ø300 wheel left hand
4		Wheel bracket (not included)
5	P04013	Din 985 M.6 nut
6	P04043	Din 985 M.12 nut
7	P04135	Din 985 M.6x60 nut
8	P04253	Din 931 M.12x140 bolt



Pos	Item number	Description
9	P04270	Din 931 M.6x75 bolt
10	P02293	Side panels
11	P03167	Socket
12	P03168	Socket
13	P03078	Socket
14	P05634N	Backrest tube with drum brake
15	P01024	Handle (not included)

Breezy 300 - Drum brakes for 12" wheel from 07/05/2012 (no. 30012050005)

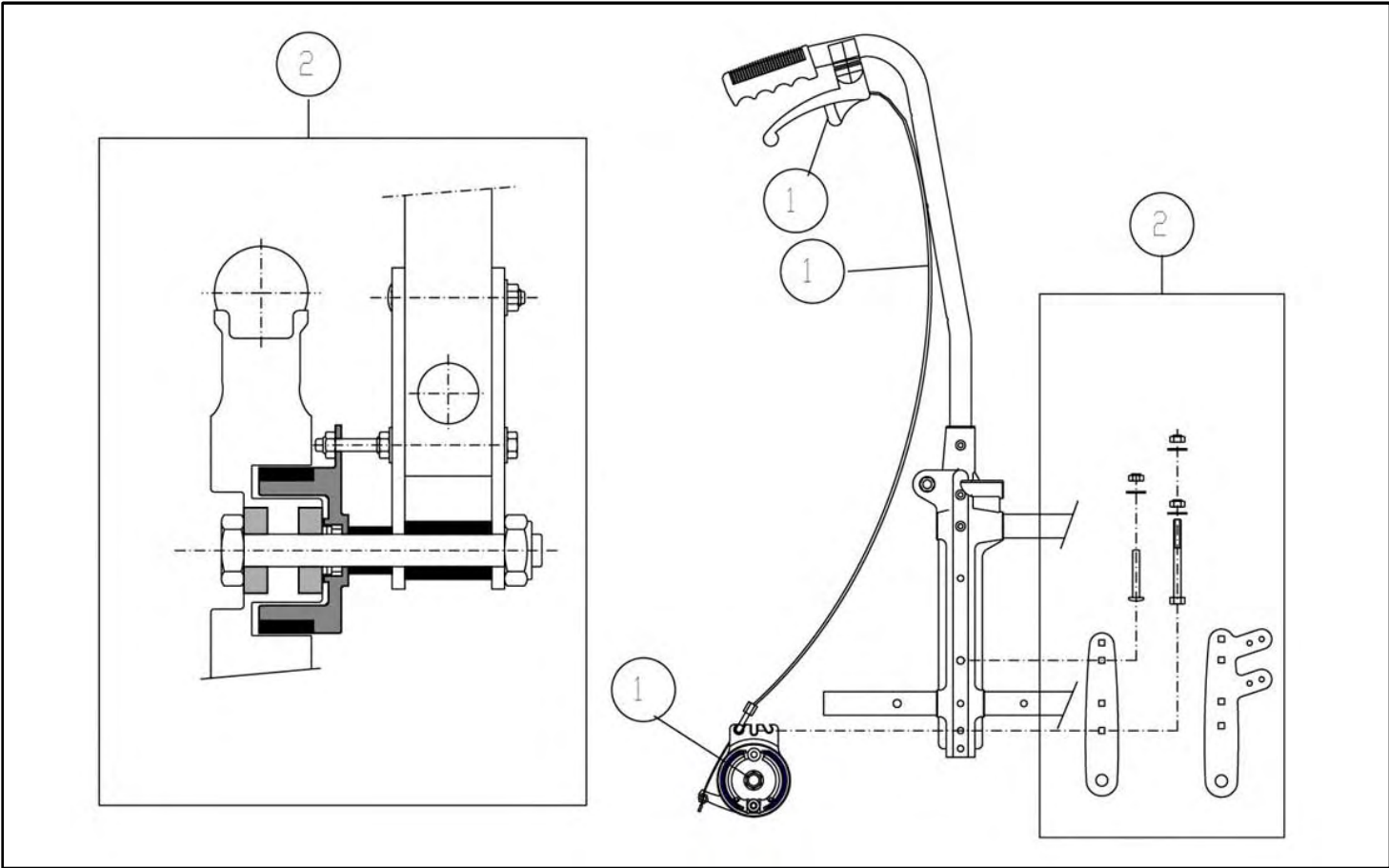


Pos	Item number	Description
1	C04090	Kit with brake shoe, cable and cover for 12" wheel - RH
1	C04091	Kit with brake shoe, cable and cover for 12" wheel - LH
2	C04121	Kit for 12" wheel and ties

Breezy 300 - Drum brakes for 24" wheel

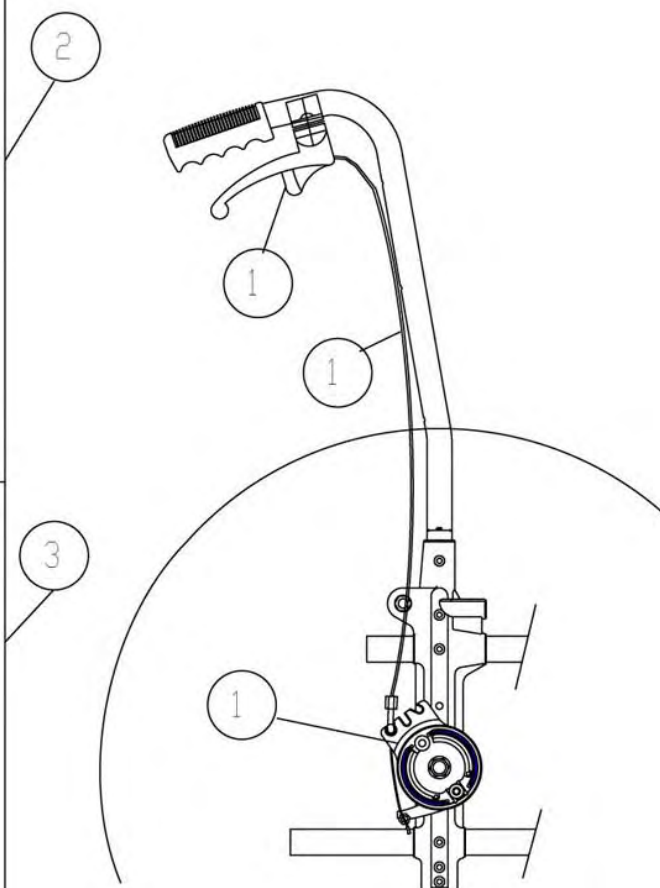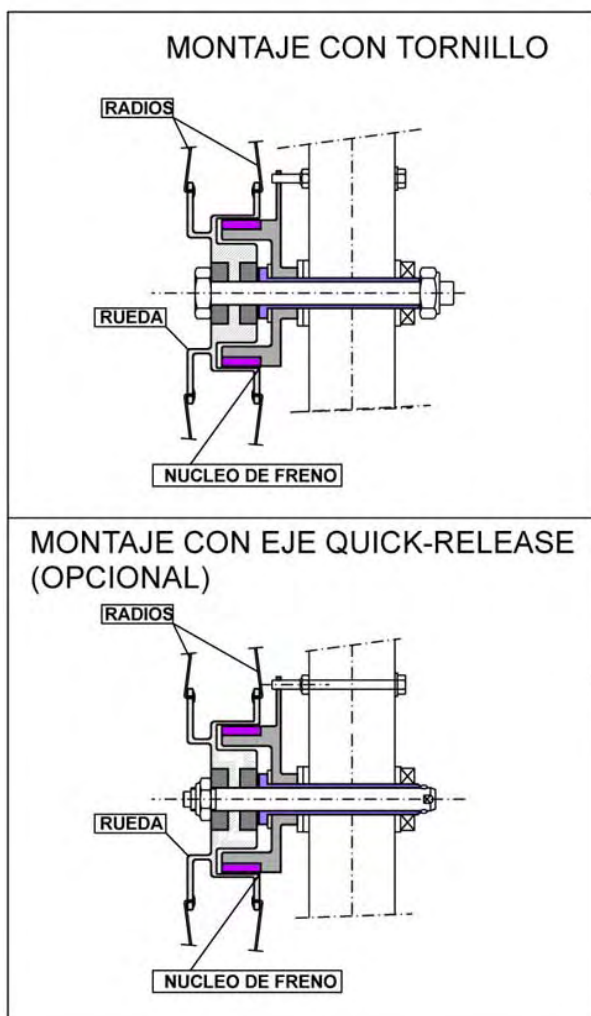


Pos	Item number	Description
1	C04093	Kit with brake shoe, cable and cover for 24" wheel - RH
1	C04094	Kit with brake shoe, cable and cover for 24" wheel - LH
2	C04097	Kit for 24" wheel and ties
3	C04098	Kit for 24" wheel and ties - Quick-release

Breezy 300 - Standard headrest

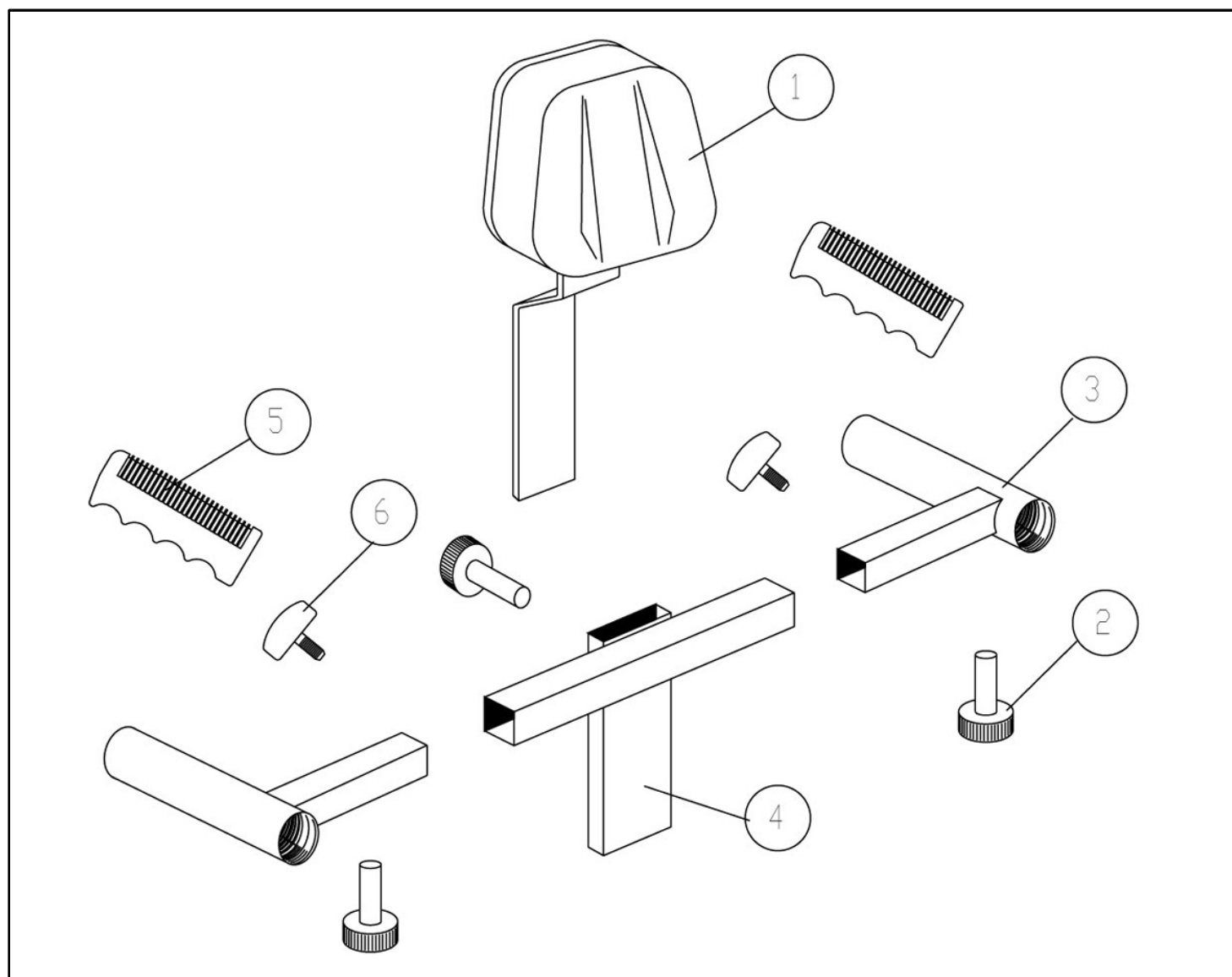


Pos	Item number	Description
1-6	C02072	Complete assembly - standard headrest
1	C02052	Upholstered headrest
2	P01254	M.8 x 35 bolt head
3	C01804 N	Mounting arm - RH.
3	C01805 N	Mounting arm - LH.
4	C01843N	Stiffener tube
5	P01606	Handle grip
6	P01604	M.8 x 16 bolt head

Breezy 300 - Standard headrest for anatomic backrest

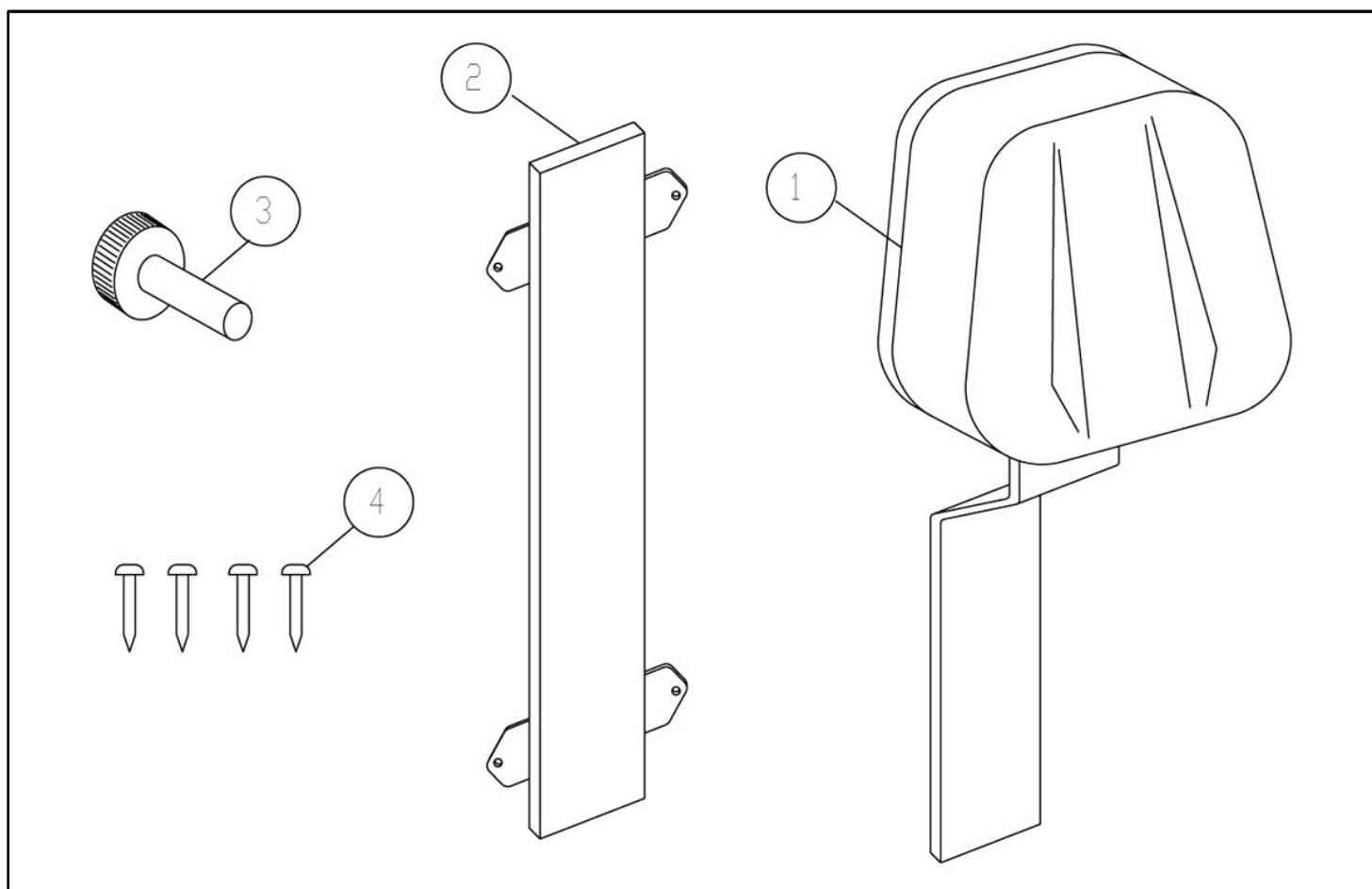


Pos	Item number	Description
1-4	C02074	Complete assembly - standard headrest for anatomic backrest
1	C02052	Upholstered headrest
2	C01822N	Vertical support
3	P01254	M.8 x 35 bolt head
4	P04083	Bolt

Breezy 300 - Standard headrests for reclining chairs

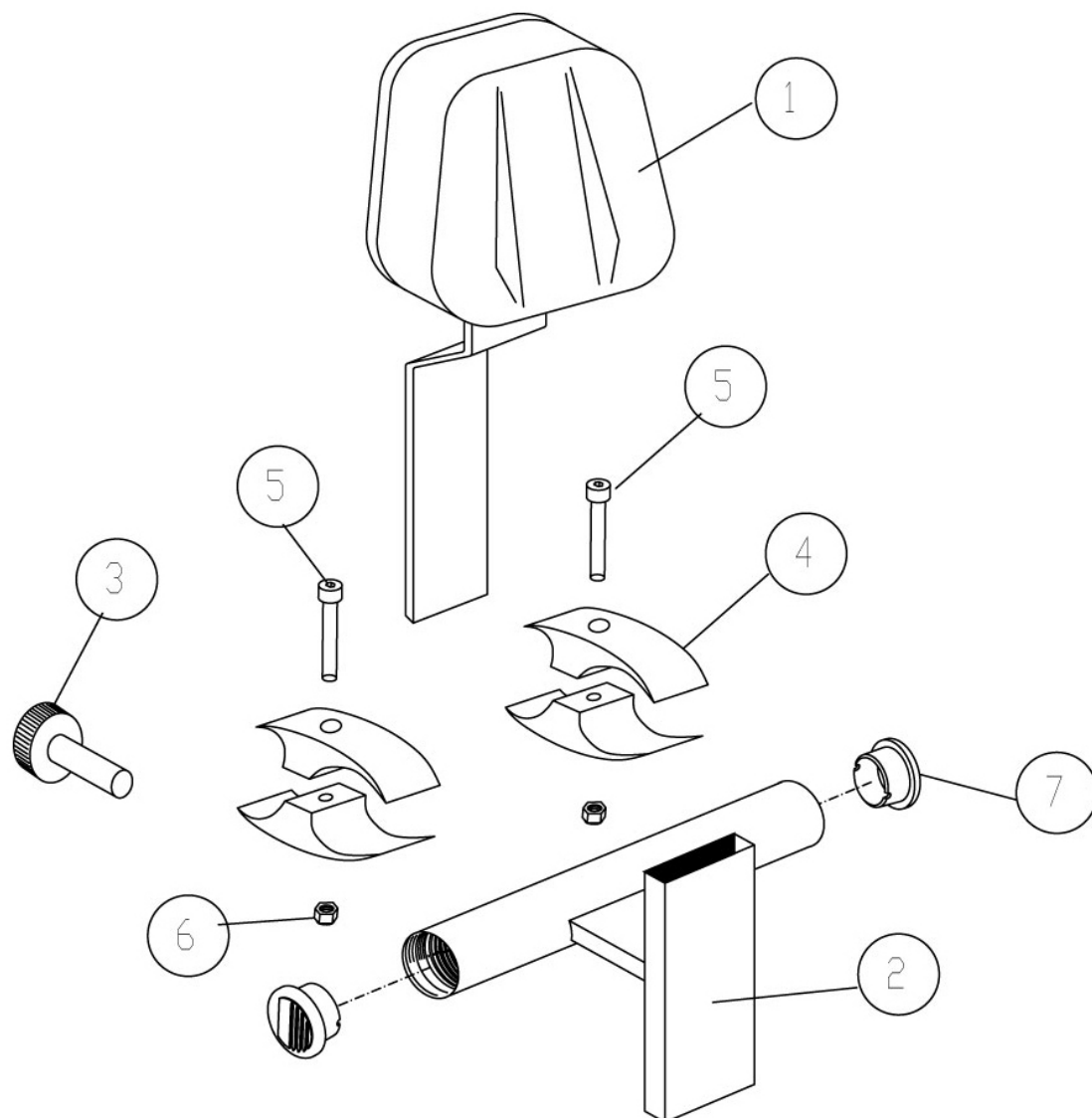


Pos	Item number	Description
1-7	C02606	Standard headrests for reclining chairs, complete assembly
1	C02052	Upholstered headrest
2	C01839N	Headrest support stiffener
3	P01254	M.8 x 35 bolt head
4	P11074	Clamp
5	P04216	Din 912 M.6 x 35 bolt
6	P04013	Din 985 M.6 nut
7	P01031	Cap

Breezy 300 - Anatomic headrest

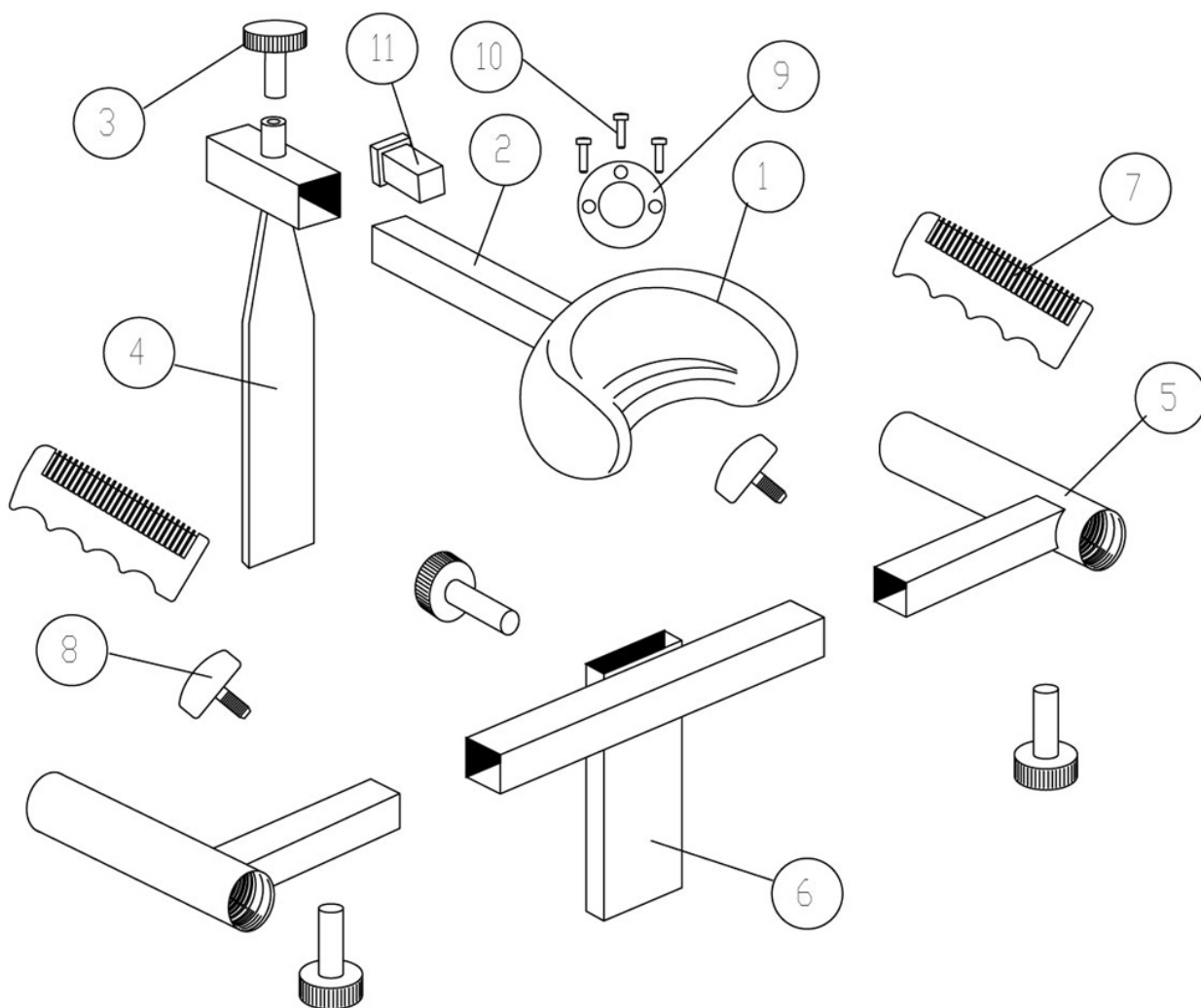


Pos	Item number	Description
1-11	C02073	Complete assembly - anatomic headrest
1	C02071	Anatomic headrest PU-KO-6-S
2	C01110C	Adjustable headrest support
3	P01254	M.8 x 35 bolt head
4	C01841N	Vertical support
5	C01804N	Mounting arm - RH.
5	C01805N	Mounting arm - LH.
6	C01843N	Stiffener tube
7	P01606	Handle grip
8	P01604	M./8 x 16 bolt head
9	P02097	Spherical cover
10	P04212	Din 7985 M.5 x 20 bolt
11	P01046	Cap 12 x 12

Breezy 300 - Anatomic headrest for anatomic backrest

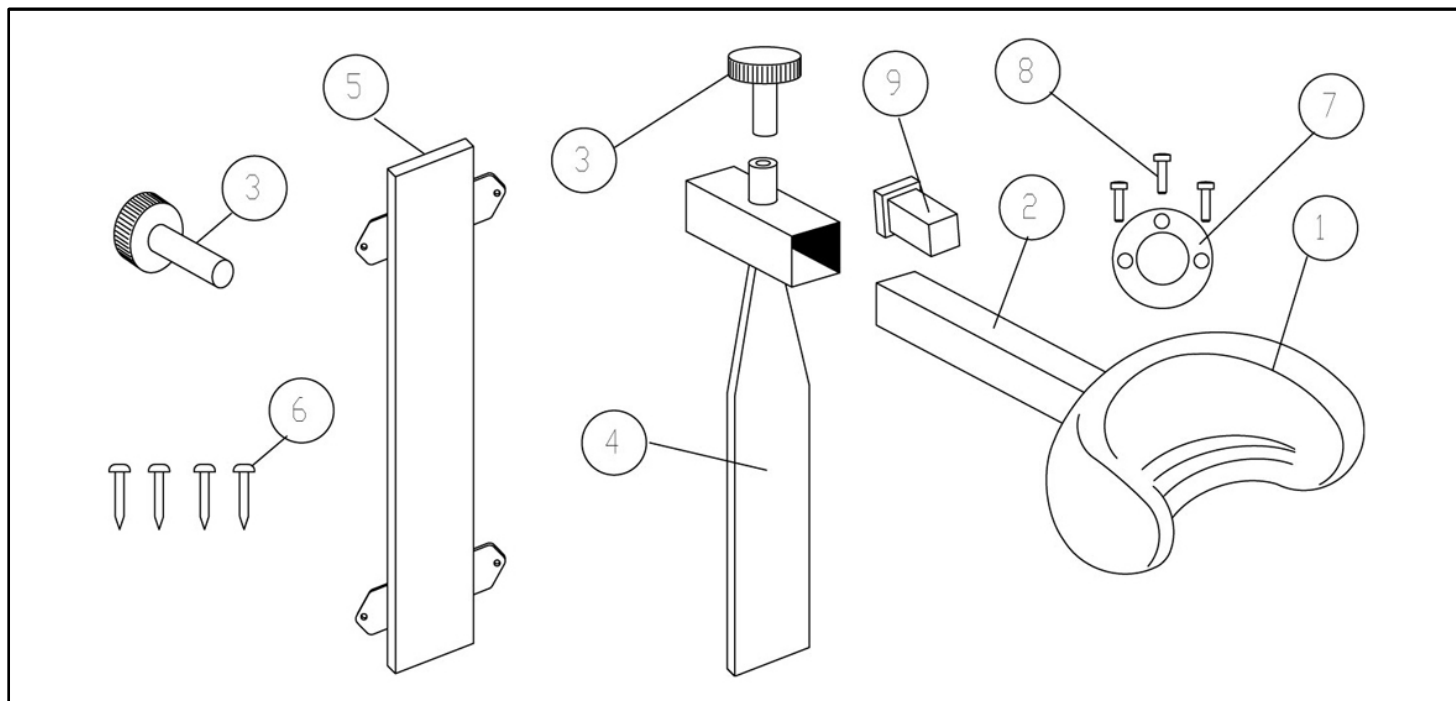



Pos	Item number	Description
1-9	C02075	Complete assembly - anatomic headrest for anatomic backrest
1	C02071	Anatomic headrest PU-KO-6-S
2	C01110C	Adjustable headrest support
3	P01254	M.8 x 35 bolt head
4	C01841N	Vertical support
5	C01822N	Vertical support
6	P04059	Bolt
7	P02097	Spherical cover
8	P04212	Din 7985 M.5 x 20 bolt
9	P01046	Cap 12 x 12

Breezy 300 - Anatomic headrests for reclining chairs

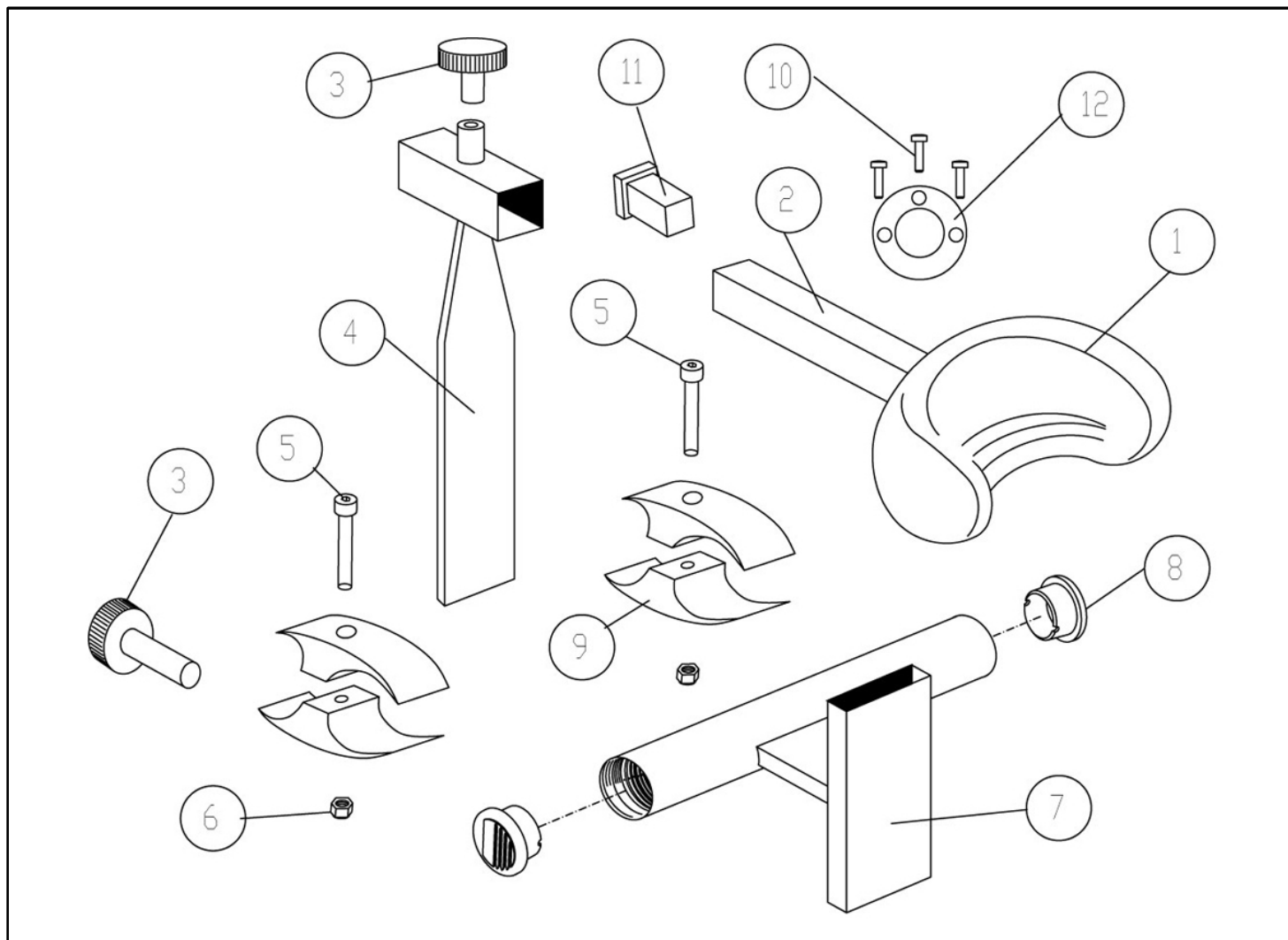


Pos	Item number	Description
1-12	C02452	Anatomic headrests for reclining chairs, complete assembly
1	C02071	Upholstered headrest
2	C01110C	Adjustable headrest support
3	P01254	M.8 x 35 bolt head
4	C01841N	Vertical support
5	P04216	Din 912 M.6 x 35 bolt
6	P04013	Din 985 M.6 nut
7	C01763N	Headrest support stiffener
8	P01031	Cap
9	P11074	Clamp
10	P04212	Din 7985 M.5 x 20 bolt
11	P01046	12x12 plug
12	P02097	Spherical cover

Breezy 300 - Back Extension (for reclining backrest)

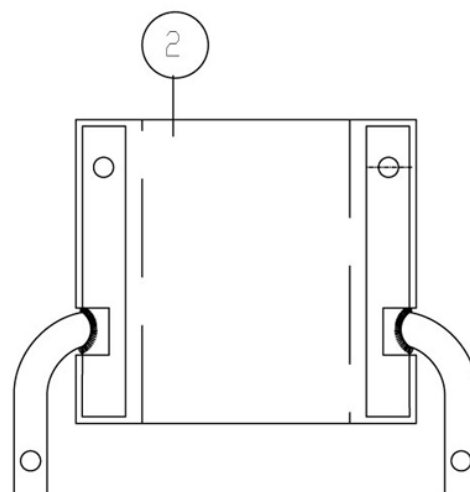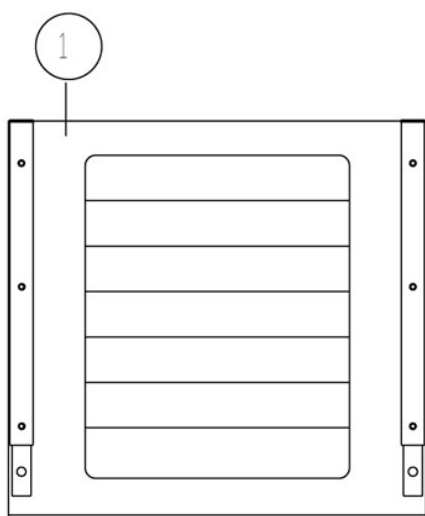


Pos	Item number	Description
1	C02560	Back extension assembly WELDED NYLON (skay) 37 cm, GREY
1	C02561	Back extension assembly WELDED NYLON (skay) 40 cm, GREY
1	C02562	Back extension assembly WELDED NYLON (skay) 43 cm, GREY
1	C02563	Back extension assembly WELDED NYLON (skay) 46 cm, GREY
1	C02564	Back extension assembly WELDED NYLON (skay) 49 cm, GREY
1	C02565	Back extension assembly WELDED NYLON (skay) 52 cm, GREY
2	C02724	Back extension assembly NYLON 37 cm, BLACK
2	C02725	Back extension assembly NYLON 40 cm, BLACK
2	C02726	Back extension assembly NYLON 43 cm, BLACK
2	C02727	Back extension assembly NYLON 46 cm, BLACK
2	C02728	Back extension assembly NYLON 49 cm, BLACK
2	C02729	Back extension assembly NYLON 52 cm, BLACK

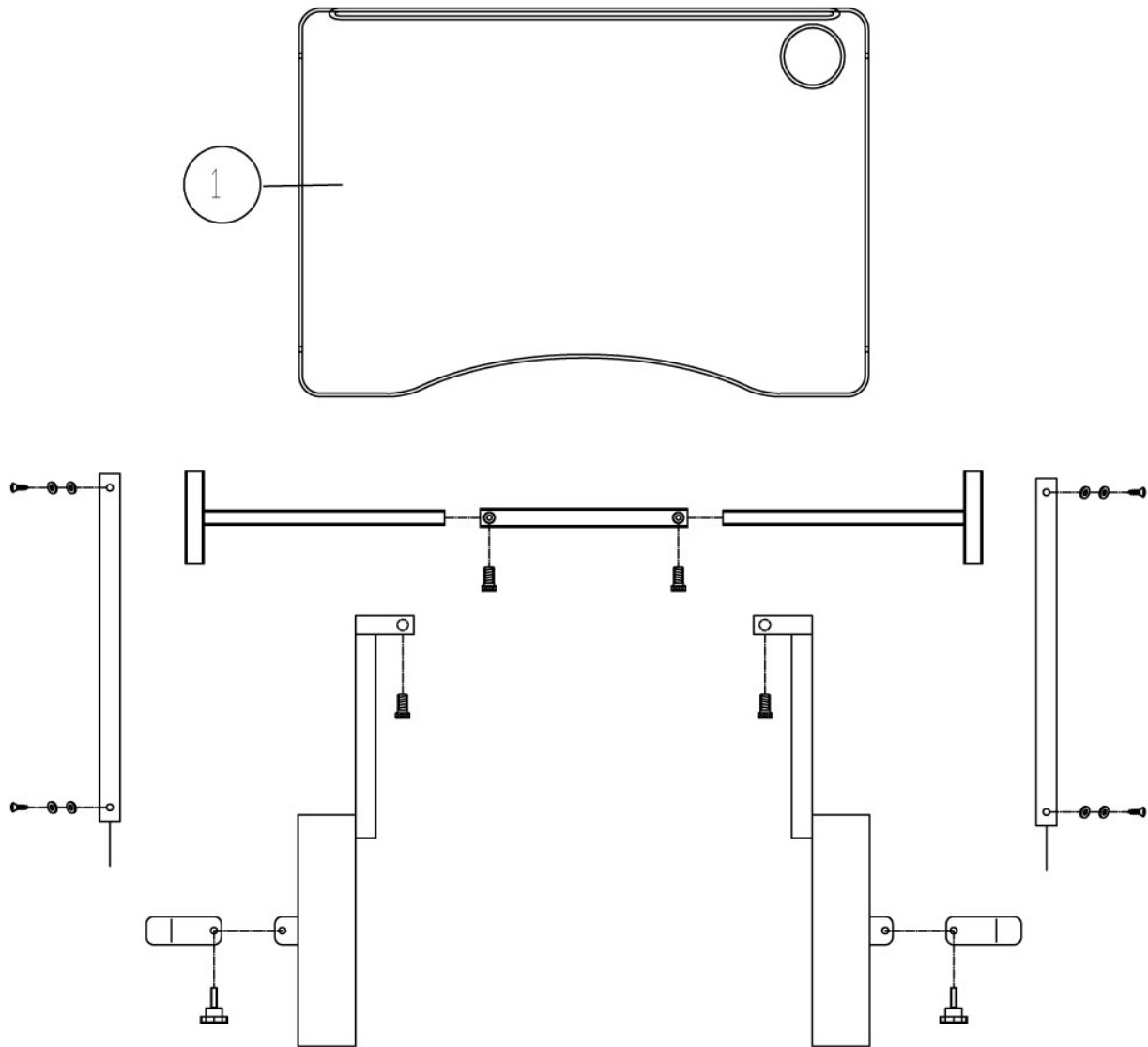
Breezy 300 - Eating and Reading Table

Pos	Item number	Description
s/n	C08272	Tray assembly
1	P01222	Table tray

Breezy 300 - Drip support

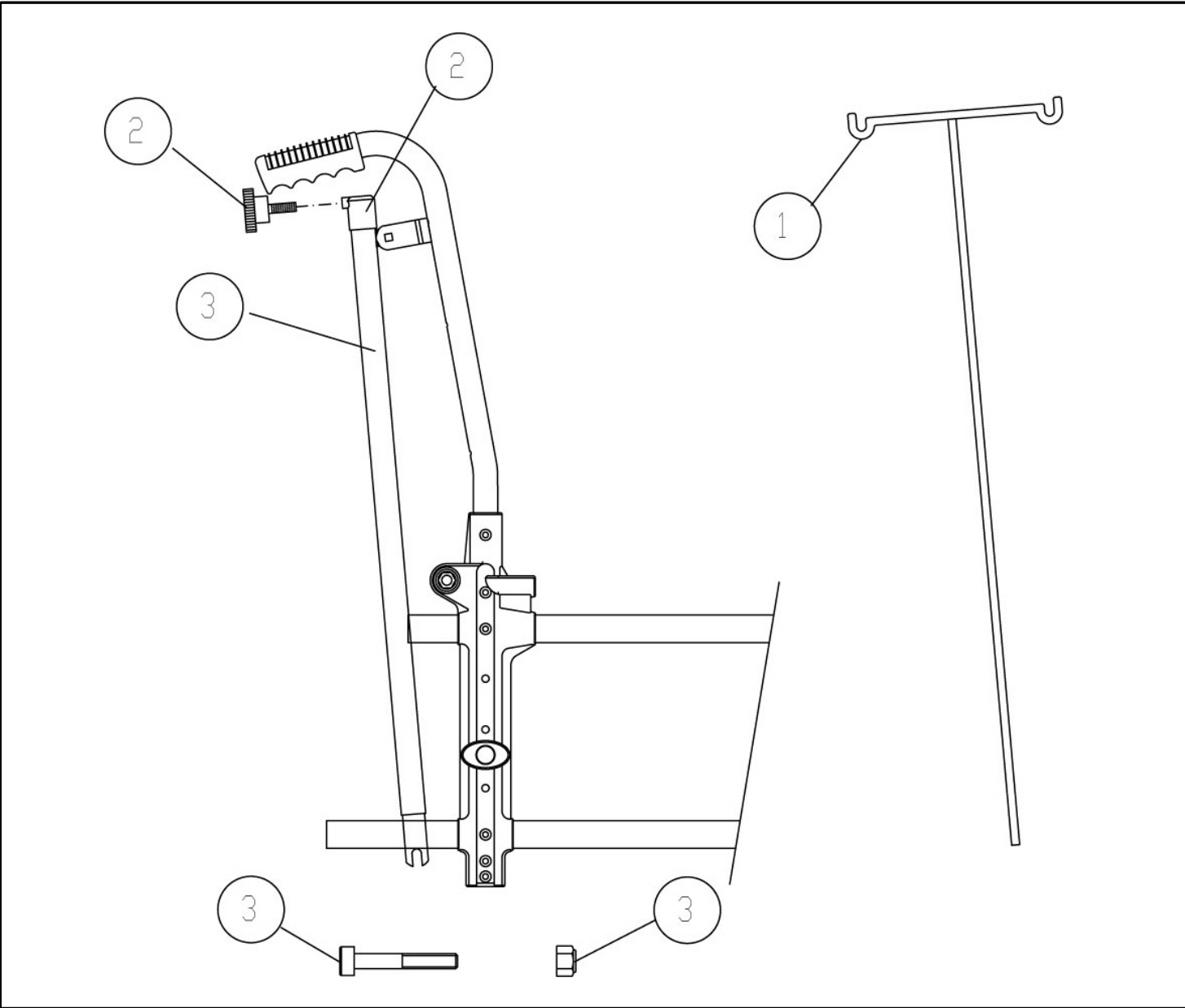


Pos	Item number	Description
1	C08061	Drip holder
2	C04085	M8x35 Bolt Head
3	C04086	Drip guide nut holder

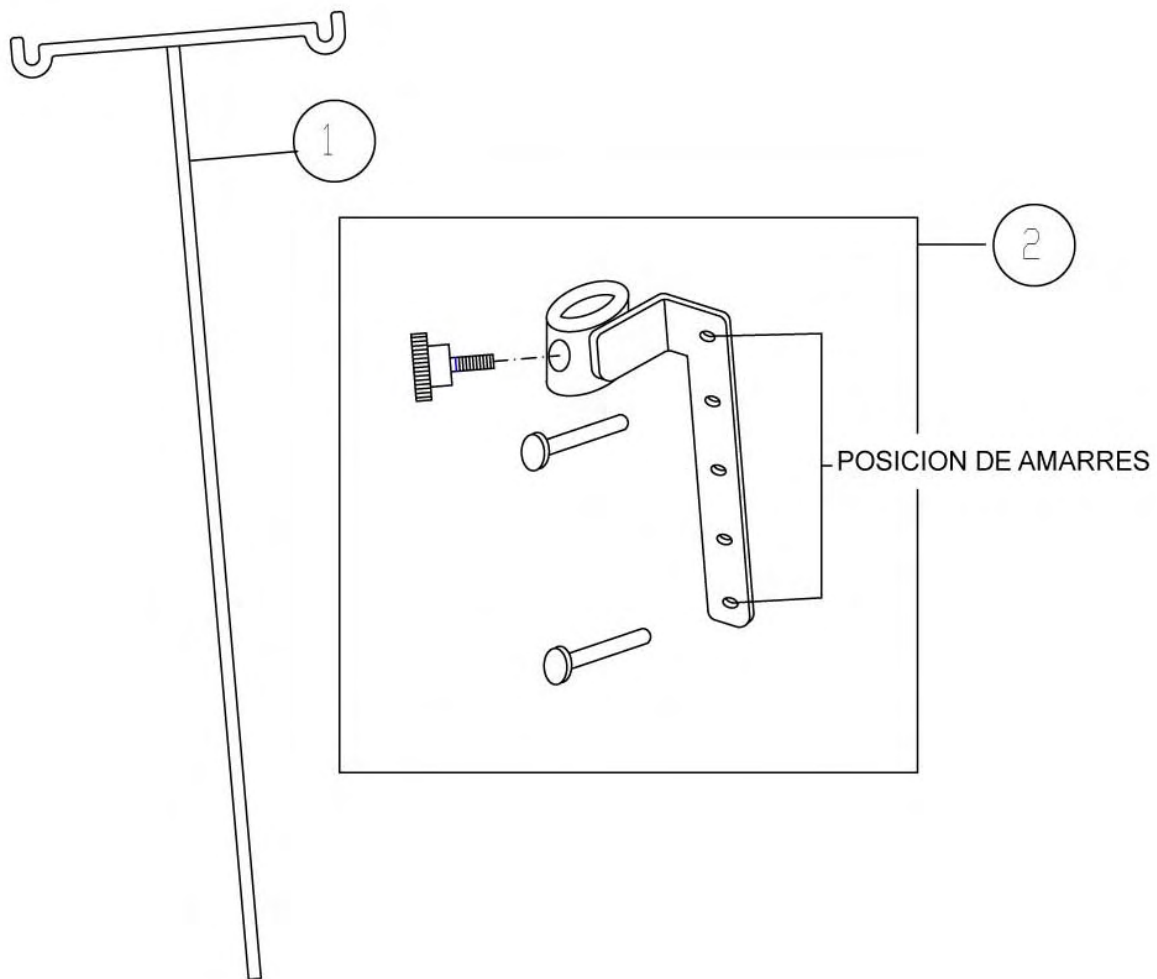
Breezy 300 - Basic drip support

Pos	Item number	Description
1	C08061	Drip holder
2	C04082	Plate axle assembly

Breezy 300 - IV Support with multiposition clamp

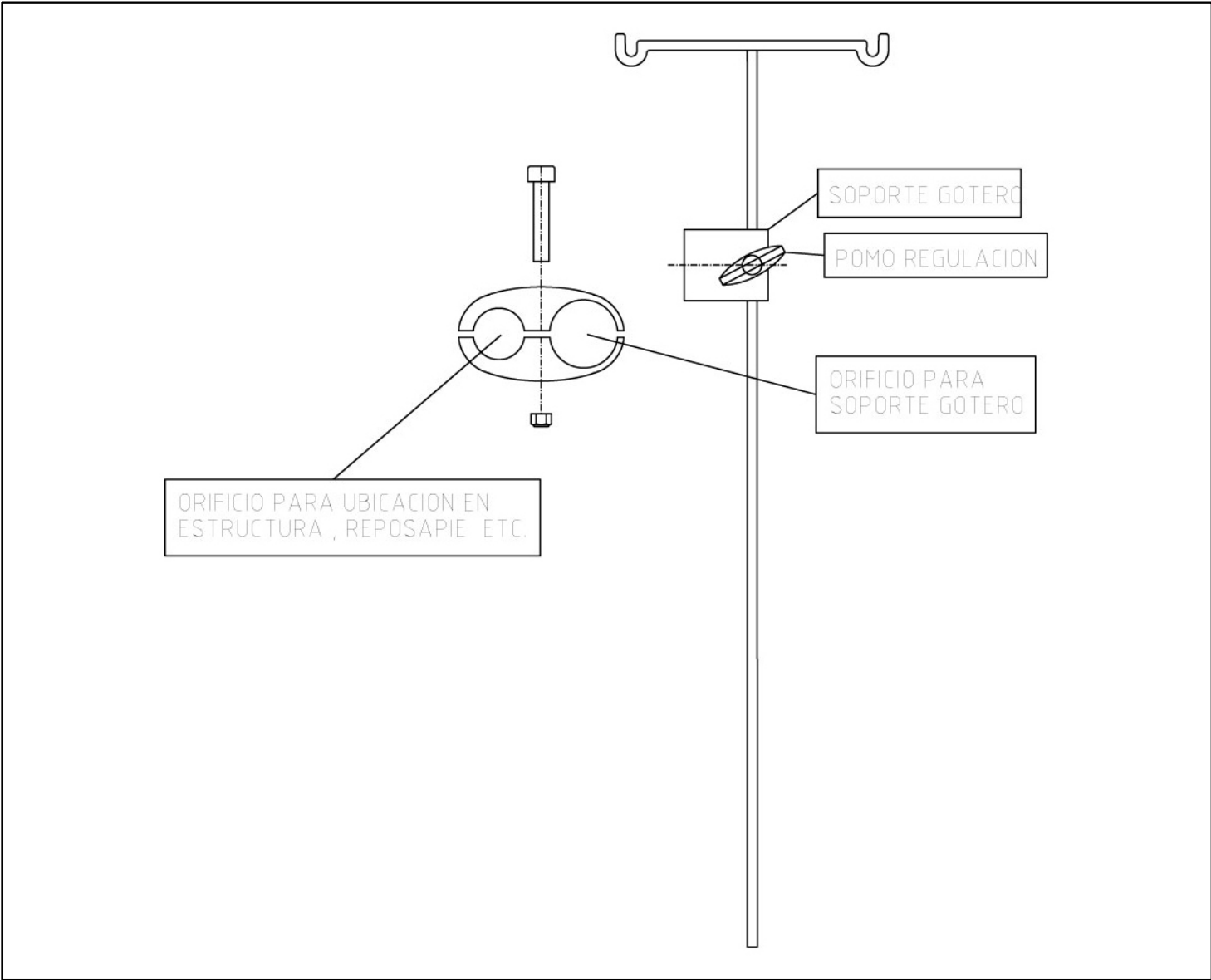


Pos	Item number	Description
s/n	C04106	Complete IV Support with multiposition clamp

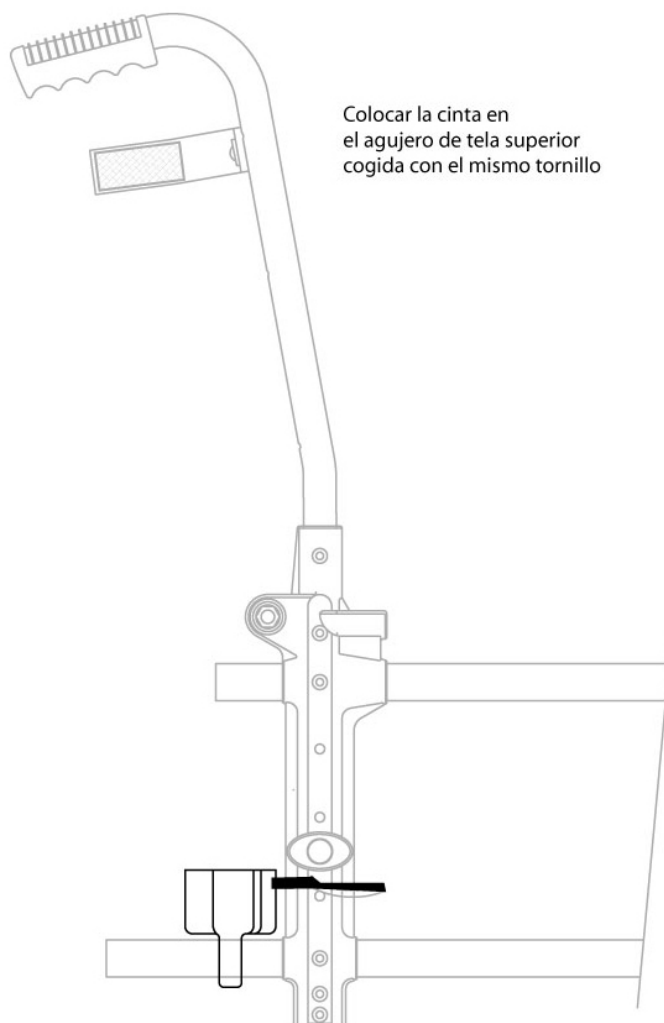Breezy 300 - Crutch Holder



Pos	Item number	Description
s/n	C01381	Crutch holder assembly

Breezy 300 - Tank holder

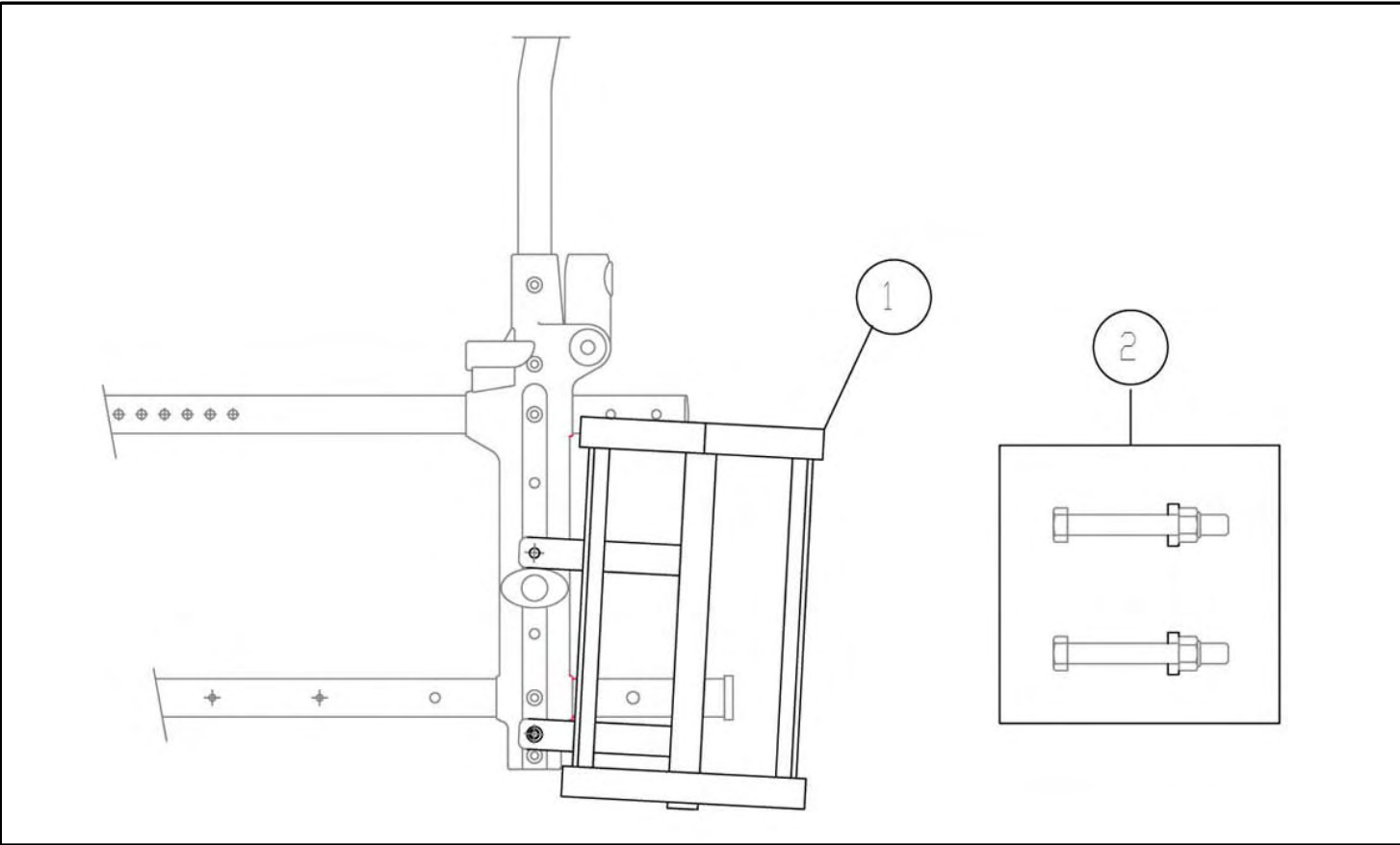


Pos	Item number	Description
1	C08273N	Tank holder
2	C011022	Bolts kit for 24" wheel
3	C011023	Bolts kit for 12" wheel

Breezy 300 - Double Amputee Support

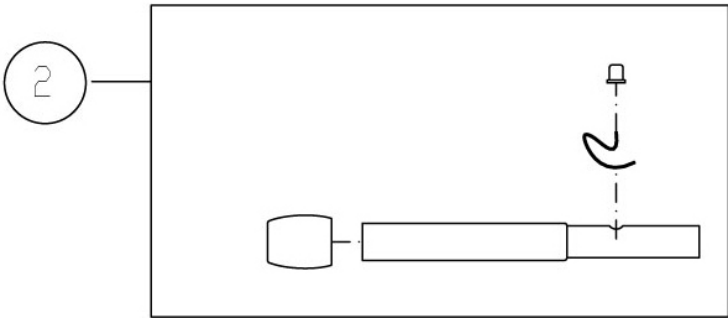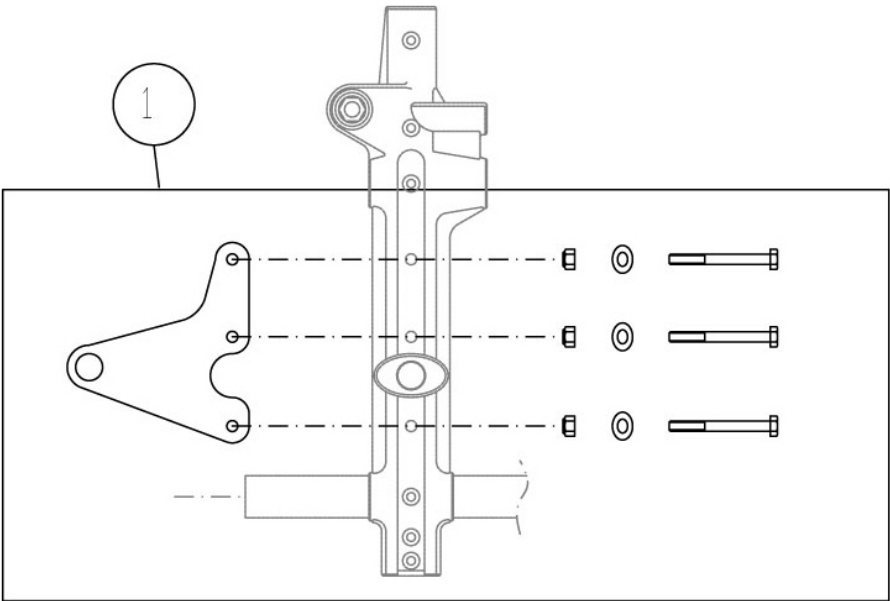


Pos	Item number	Description
1	C011024	Double amputee support kit
2	C04084	Tipping tube, assembly

Breezy 300 - Transit wheel

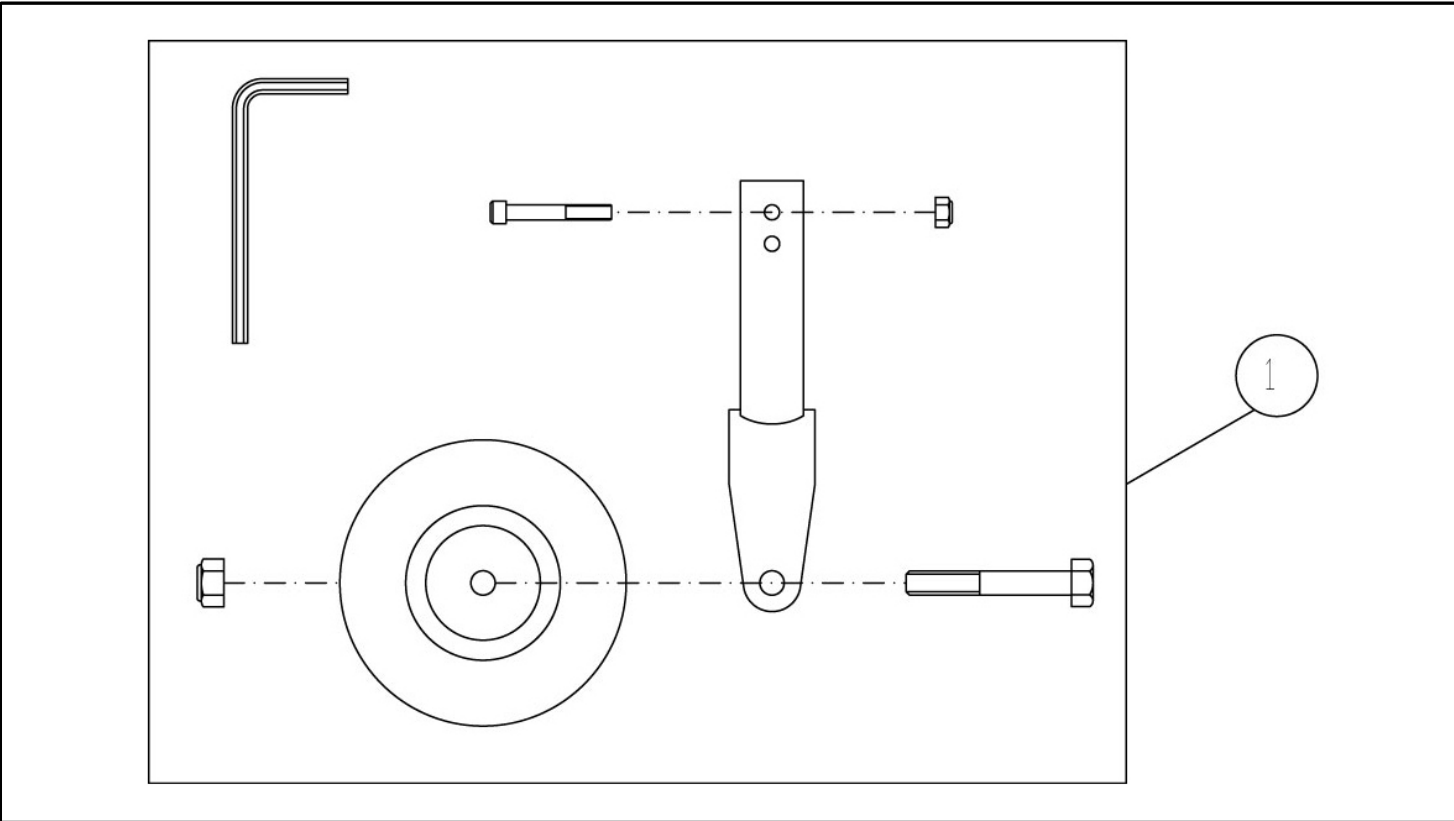


Pos	Item number	Description
1	C06105	Transit wheel assembly

Breezy 300 - Anti-Tip Tubes

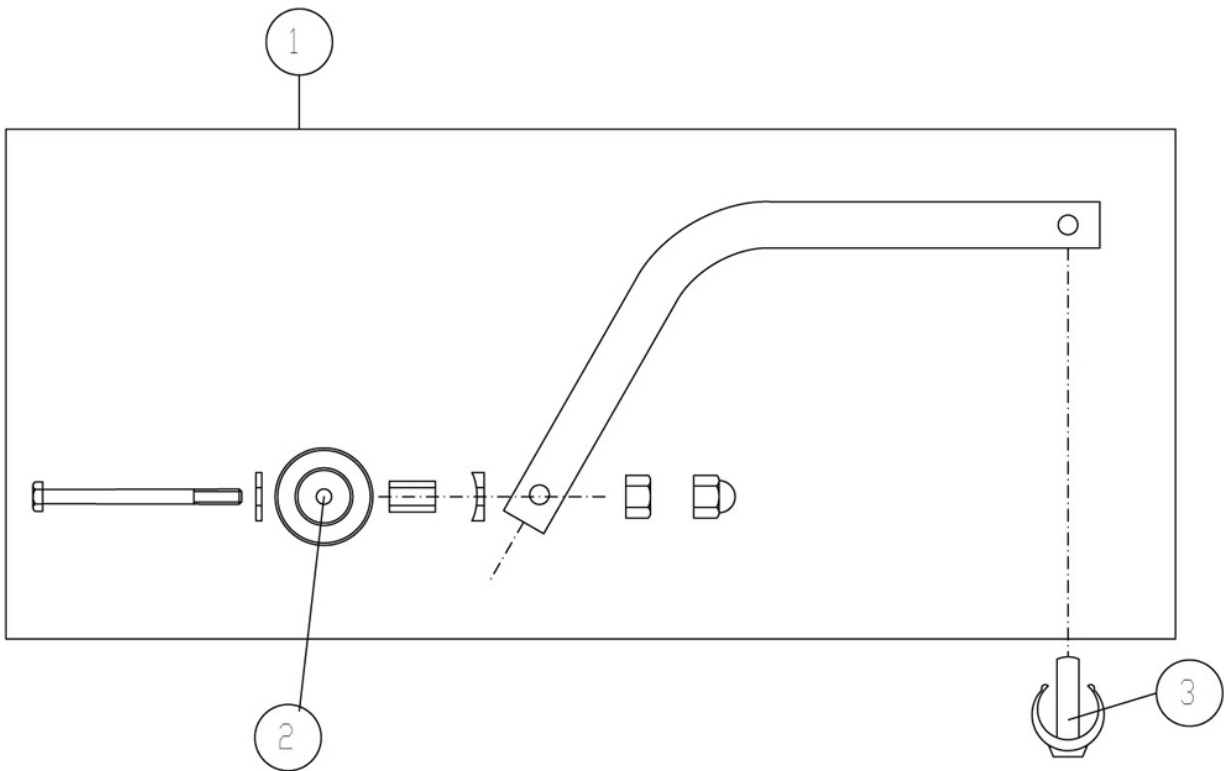

Pos	Item number	Description
1	C06240	Pair of anti-tip tubes, complete assembly
1	C06236	Anti-tip tube assembly, right
1	C06237	Anti-tip tube assembly, left
1	C06241	Pair of anti-tip tubes, complete assembly for RECLINING CHAIR
1	C06238	Anti-tip tube assembly, right, for RECLINING CHAIR
1	C06239	Anti-tip tube assembly, left, for RECLINING CHAIR
2	P01122	Wheel
3	C01673	Clip D25 O 8


Breezy 300 - Unfolding device

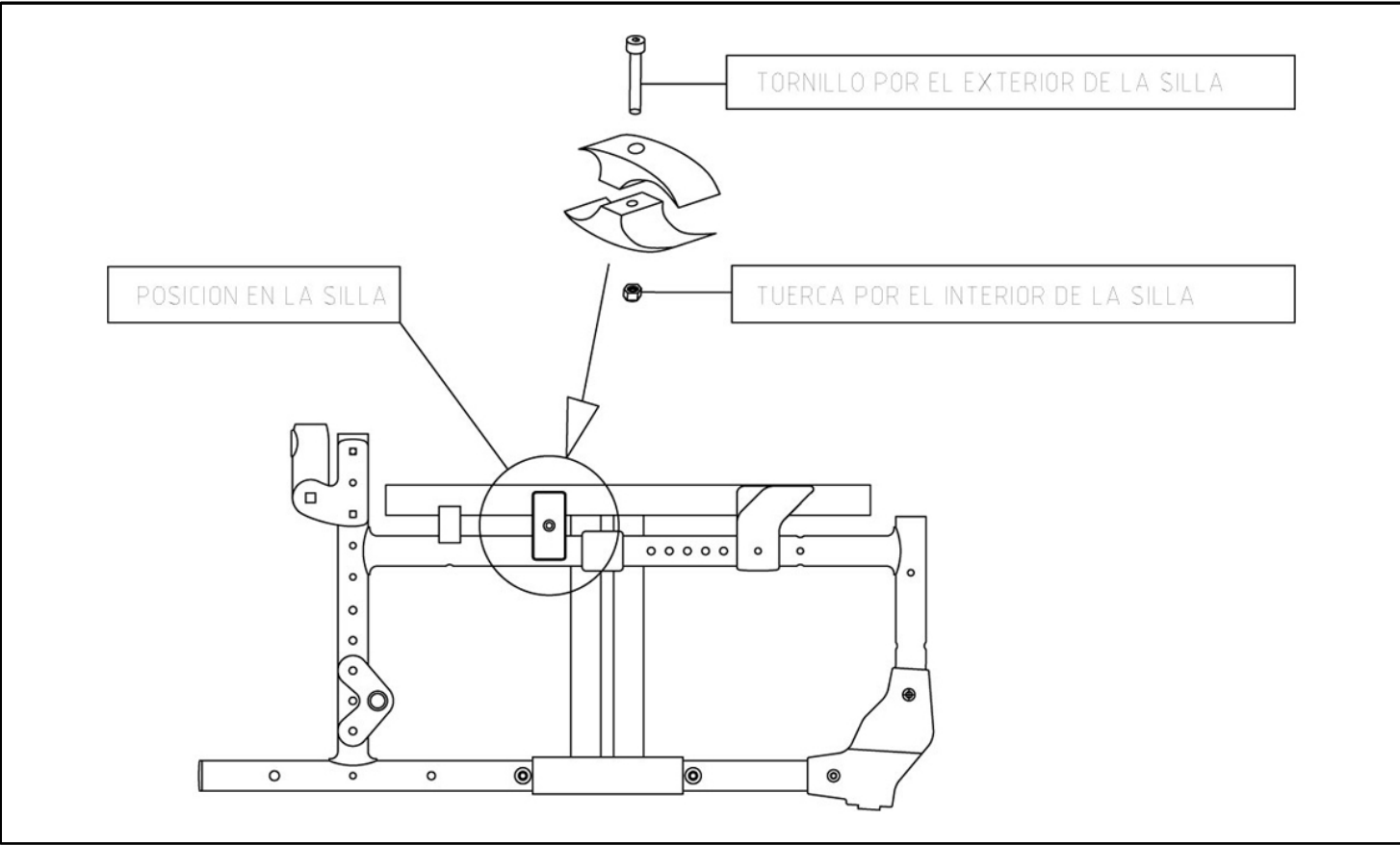

Pos	Item number	Description
s/n	C04105	Unfolding device

Breezy 300 - Transportation kit

CHAPAS COLOCADAS EN LAS PARTES INTERIORES DE LAS SILLAS

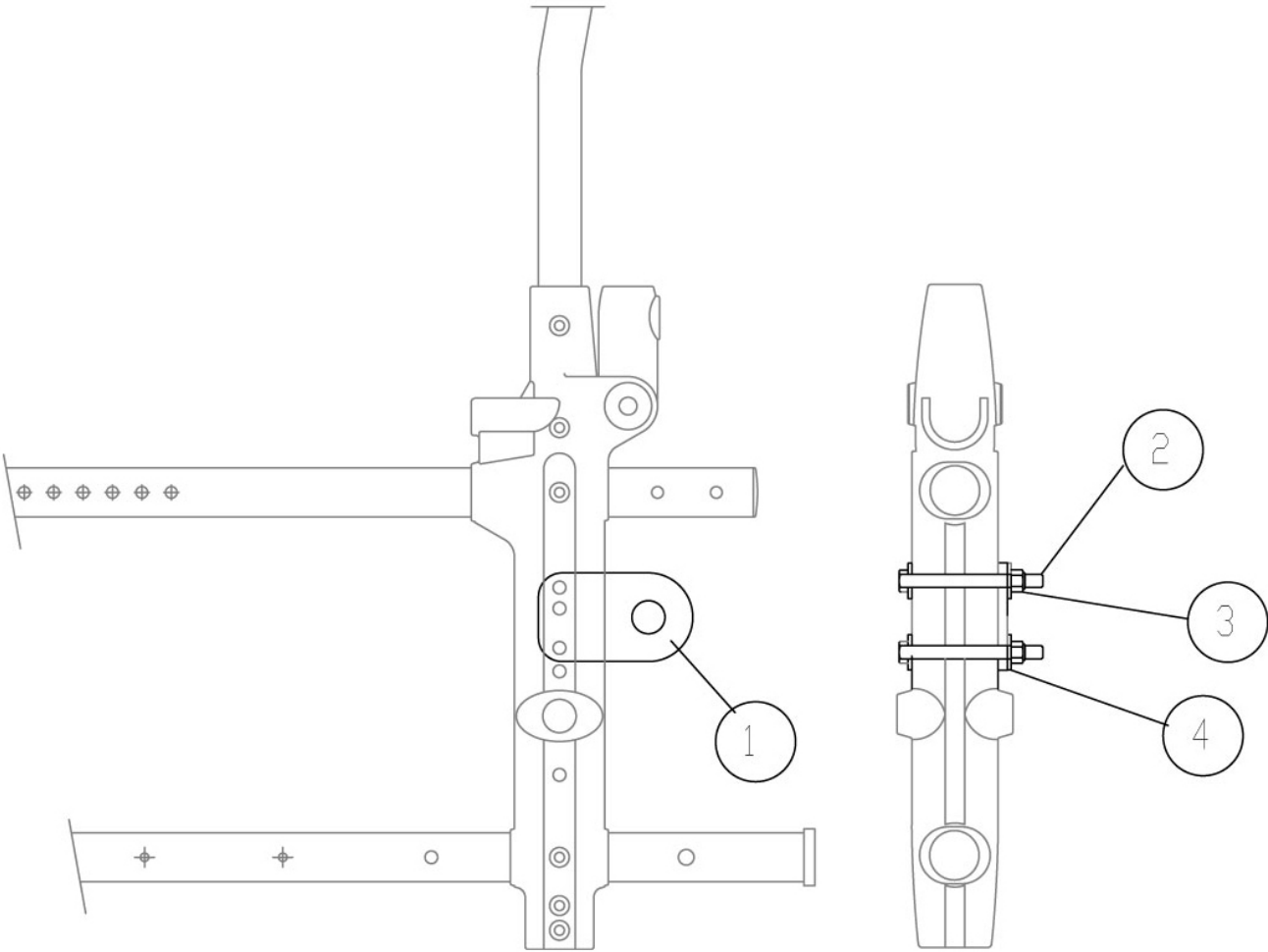

Pos	Item number	Description
s/n	C04131	Complete transportation kit