

Model No: 10381C

For Use With: 10380C Merlin Raised Toilet Seat Frame

10399C Aluminium Toilet Surround

10397C Steel Toilet Surround

These floor fixing feet are designed to be used in conjunction with toilet surrounds, which are to assist those who have difficulties lowering or raising themselves to a conventional toilet. It may also be used with a raised toilet seat.

Using the toilet surround with floor fixing feet gives extra stability and security.

WARNING

Where a person puts an uneven pressure on one arm to raise or lower themselves, care must be taken to ensure the frame does not tip sideways.

FIXING INSTRUCTIONS

1. The floor fixing feet are designed to be used with the rear legs of the toilet surround.
2. The floor fixing feet should be securely fixed to a hard surface such as floor boards etc.
3. Place the frame around the toilet as shown in Fig 1. Place the back legs into the floor fixing feet.
4. The frame can then be removed between uses if desired

INSTRUCTIONS FOR USE

To adjust the height of the toilet frame:

1. Remove the horseshoe clip which is located on the lower leg of the frame, place your thumbs at either side on the open end of the clip and pull off (Fig 2).
2. Adjust the length by extending the foot so the frame is at the desired height (Select from seven different positions), (Fig 3).
3. To replace the horseshoe clip, align the holes on leg and foot. Insert the horseshoe clip pin into the hole and push the clip on firmly until it snaps around the leg (Fig 4).
4. Repeat for the three remaining legs.
5. Ensure all legs are even and horseshoe clips are fully located.

COOPERS FLOOR FIXING FEET

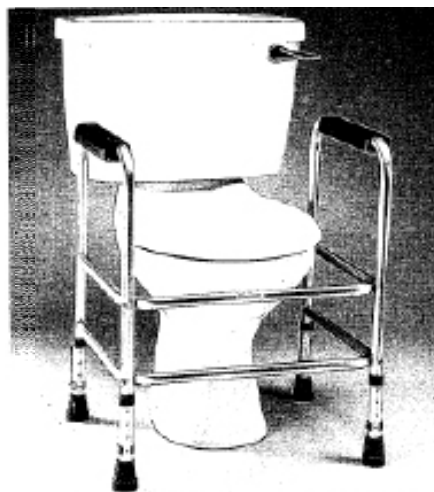


FIG 1



FIG 2



FIG 4

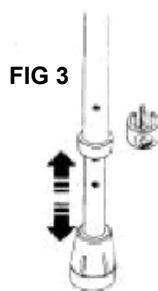


FIG 3

GENERAL SAFETY

- 1 Take care with wet floors.
- 2 All equipment should be regularly inspected for any damage or significant signs of wear.

Check for:

- Bent or damaged tubes or cracks at tube joints.
- Horseshoe clips are not loose or worn.
- Elongated adjustment holes.
- Smooth, worn or split ferrules (rubber tips).
- Replace as necessary.

Regular cleaning with a mild disinfectant is recommended. **DO NOT** use bleach or any solvents (such as lighter fluid).

LIFE EXPECTANCY:

We estimate a life expectancy of 2 years for this product, provided that:

- It is used in strict accordance with the intended use as set out in this document.
- All service and maintenance requirements are met.

The estimated life expectancy can be exceeded if the product is carefully used and properly maintained. The life expectancy can also be considerably reduced by extreme or incorrect usage.

The fact that we estimate a life expectancy for this product does not constitute an additional warranty.

WARRANTY:

All Coopers products are warranted for 12 months from the date of purchase against manufacturing or design defects.

The warranty is subject to the following conditions:

- It does not apply to wear and tear, inappropriate or incorrect use, or non-observance of User Instructions.
- Defective set up by the user or third parties or negligent or careless treatment.
- Use of unsuitable or unapproved spares or unauthorised alteration.
- The warranty does not extend to consequential costs resulting from fault clearance, loss of earnings or expenses. **THIS IN NO WAY AFFECTS YOUR STATUTORY RIGHTS.**



Sunrise Medical Limited

Thorns Road, Brierley Hill, West Midlands, DY5 2LD England
Tel + 44 (0) 845 605 6688 Fax + 44 (0) 845 605 6689



Conforms to
The Medical Device Directive 93/42/EEC 1993