



# **NEW ZIPPIE® TS**

# PARTS MANUAL

(ZTS ZTSE)  
JANUARY 2013

Sunrise Medical appreciates your business. For your ordering convenience, our Customer Service Representatives can be contacted at:

**To place orders:  
(800)333-4000**

**Quickie Technical Assistance:**

You can reach our Technical Support Team by choosing prompt 1 followed by the # sign, then prompt 2 for Technical Support. Choose option 1 for Quickie Technical Support.

**For Assistance with a Web Order,  
or to sign up for [sunrisemedical.com](http://sunrisemedical.com)  
(800) 333-4000 Option 5**

**FOR YOUR CONVENIENCE YOU MAY PLACE ORDERS THROUGH  
OUR WEBSITE OR BY FAX 24 HOURS A DAY:**

**[www.sunrisemedical.com](http://www.sunrisemedical.com)**

**Fax: (800)300-7502**

For easy reference please fill in your Customer Number and Sunrise Account Manager's name.

Customer Account Number \_\_\_\_\_  
Ship to Account Number \_\_\_\_\_  
Sunrise Account Manager \_\_\_\_\_


\*Part prices and configurations or specifications of products included in this Parts Manual are subject to change without prior notice.

|                                                                                         |    |
|-----------------------------------------------------------------------------------------|----|
| ZIPPIE TS / NEW ZIPPIE TS / ZIPPIE TS SE (Chairs w/ S/N Prefix ZP, ZT, ZTS, & ZTE)..... | 6  |
| BASE FRAME .....                                                                        | 6  |
| FRAME .....                                                                             | 6  |
| ZIPPIE TS FRAME (Chairs w/ S/N Prefix ZP).....                                          | 6  |
| FRAME (FIXED FRONT).....                                                                | 6  |
| FRAME (FIXED FRONT) DISC. 11/16/99 .....                                                | 8  |
| FRAME (SWING-AWAY).....                                                                 | 10 |
| FRAME (SWING-AWAY) DISC. 11/16/99 .....                                                 | 12 |
| LOCKING MECHANISM .....                                                                 | 14 |
| LOCKING MECHANISM DISC. 11/16/99.....                                                   | 16 |
| NEW ZIPPIE TS FRAME (Chairs w/ S/N Prefix ZT, ZTS & ZTSE) .....                         | 17 |
| FRAME (RIGID).....                                                                      | 18 |
| FRAME (FOLDING).....                                                                    | 20 |
| CROSS TUBE ASSM.....                                                                    | 22 |
| LOCKING MECHANISM NEW ZIPPIE TS .....                                                   | 24 |
| TRANSIT KIT ZIPPIE TS.....                                                              | 26 |
| AXLES & AXLE PLATES .....                                                               | 26 |
| AXLE BRACKET .....                                                                      | 27 |
| AXLE BRACKET 12" & 16" .....                                                            | 28 |
| QUICK RELEASE AXLES .....                                                               | 29 |
| CASTERS & FORKS.....                                                                    | 29 |
| FORK & STEM ASSEMBLIES .....                                                            | 30 |
| CASTER STEMS.....                                                                       | 31 |
| CASTER PIN LOCK .....                                                                   | 32 |
| CASTER PLATE.....                                                                       | 33 |
| CASTER TIRES & TUBES .....                                                              | 34 |
| CASTERS .....                                                                           | 34 |
| MICRO CASTERS.....                                                                      | 35 |
| 3 SPOKE CASTERS .....                                                                   | 36 |
| SOFT ROLL CASTERS.....                                                                  | 37 |
| 8" x 2" CASTERS .....                                                                   | 38 |
| ANTI-TIPS .....                                                                         | 38 |
| ANTI TIPS .....                                                                         | 39 |
| FRONT ANTI-TIP .....                                                                    | 41 |
| ANTI-TIPS (FITS 12" & 16" REAR WHEELS).....                                             | 42 |
| WHEEL LOCKS .....                                                                       | 42 |
| HIGH WHEEL LOCK PUSH (LONG SLIDE).....                                                  | 43 |
| HIGH WHEEL LOCK PULL (LONG SLIDE) .....                                                 | 45 |
| FOOT LOCK 12" WHEEL.....                                                                | 47 |
| HIGH WHEEL LOCK EXTENSION HANDLE (QUICKIE) .....                                        | 49 |
| GRADE AID ONLY ASSEMBLY.....                                                            | 51 |
| Z-LOCK .....                                                                            | 53 |
| WHEEL LOCK PUSH TO LOCK (QUICKIE STYLE).....                                            | 54 |
| WHEEL LOCK PULL TO LOCK (QUICKIE STYLE).....                                            | 56 |
| WHEELS & TIRES .....                                                                    | 57 |
| WHEELS .....                                                                            | 57 |
| 12" MAG .....                                                                           | 58 |
| 16" MAG .....                                                                           | 59 |
| MAG .....                                                                               | 60 |
| LITE SPOKE.....                                                                         | 62 |
| SPINERGY SPOX (18 SPOKE) .....                                                          | 63 |
| TIRES.....                                                                              | 63 |
| PNEUMATIC & PNEU. AIRLESS TIRE.....                                                     | 64 |
| FULL POLY TIRE .....                                                                    | 65 |
| LOW POLY TIRE.....                                                                      | 66 |
| HIGH PRESSURE CLINCHER TIRE .....                                                       | 67 |
| KEVLAR TIRE .....                                                                       | 68 |
| TUBES .....                                                                             | 68 |
| TUBES AND AIRLESS INSERTS .....                                                         | 69 |
| SEATING & POSITIONING.....                                                              | 70 |
| SEAT.....                                                                               | 70 |
| GROWING SEAT PAN.....                                                                   | 71 |
| SOLID SEAT & SOLID BACK .....                                                           | 73 |
| BACK.....                                                                               | 76 |
| BACK FRAME .....                                                                        | 76 |
| BACKREST ZIPPIE TS/2 .....                                                              | 76 |
| SWING-AWAY ADJ STROLLER HANDLE .....                                                    | 78 |
| SWING-AWAY ADJ STROLLER HANDLE RETRO KIT .....                                          | 80 |
| STROLLER HANDLE EXTENSION-REMOVABLE.....                                                | 82 |
| RIGID-IZER .....                                                                        | 84 |

|                                                     |     |
|-----------------------------------------------------|-----|
| HANGERS & FOOTPLATES.....                           | 84  |
| SWING AWAY EXT MOUNT HANGERS.....                   | 84  |
| SWING AWAY EXT MOUNT HANGERS.....                   | 84  |
| 60 DEG S/A HANGER.....                              | 85  |
| 60 DEG S/A HANGER (DISC.8-13-01) .....              | 87  |
| 60 DEG S/A HANGER (DISC.10-23-99) .....             | 88  |
| 70 DEG S/A HANGER.....                              | 89  |
| EXTENSION TUBES.....                                | 89  |
| DOUBLE D FOOTREST EXTENSION TUBE "OLD STYLE" .....  | 90  |
| HIGH MOUNT EXTENSION TUBE .....                     | 92  |
| FOOTPLATES.....                                     | 93  |
| COMPOSITE FOOTPLATE.....                            | 94  |
| ADULT ANGLE-ADJUSTABLE FOOTPLATE EXT MOUNT.....     | 96  |
| KIDS ANGLE ADJ FOOTPLATE ASSM-EXTENSION MOUNT ..... | 98  |
| PLATFORM FLIP-UP FOOTPLATE.....                     | 100 |
| FRONT MOUNT HANGERS .....                           | 101 |
| FRONT MOUNT HANGERS .....                           | 101 |
| SWING AWAY FRONT MOUNT HANGERS .....                | 101 |
| 70 DEG S/A FRONT MOUNT HANGER .....                 | 102 |
| 80 DEG S/A FRONT MOUNT HANGER .....                 | 104 |
| 90 DEG S/A FRONT MOUNT HANGER .....                 | 105 |
| LIFT OFF FRONT MOUNT HANGERS.....                   | 105 |
| 70 DEG LIFT-OFF HANGER.....                         | 106 |
| 80 DEG LIFT-OFF HANGER.....                         | 107 |
| 90 DEG LIFT-OFF HANGER.....                         | 108 |
| EXTENSIONS .....                                    | 108 |
| FRONT MOUNT HANGER EXT.....                         | 109 |
| FOOTPLATES.....                                     | 109 |
| KIDS ANGLE ADJ FOOTPLATE FRONT MOUNT .....          | 110 |
| ADULT ANGLE-ADJ FOOTPLATE FRONT MOUNT.....          | 112 |
| 90 DEG FOOTBOARD.....                               | 114 |
| ELRS .....                                          | 115 |
| EXT MOUNT ELRS.....                                 | 115 |
| EXT MOUNT ELR HANGERS.....                          | 115 |
| ELEVATING HANGER .....                              | 116 |
| ELEVATING HANGER CALF PAD 14 -15".....              | 118 |
| ELEVATING HANGER CALF PAD 16-20".....               | 119 |
| EXTENSION TUBES.....                                | 119 |
| DOUBLE D FOOTREST EXTENSION TUBE "OLD STYLE".....   | 120 |
| FOOTPLATES.....                                     | 121 |
| ELR ALUMINUM FOOTPLATE .....                        | 122 |
| COMPOSITE FOOTPLATE.....                            | 123 |
| ADULT ANGLE-ADJUSTABLE FOOTPLATE EXT MOUNT.....     | 125 |
| KIDS ANGLE ADJ FOOTPLATE ASSM-EXTENSION MOUNT ..... | 127 |
| FRONT MOUNT ELRS.....                               | 128 |
| FRONT MOUNT ELR HANGERS .....                       | 128 |
| 90 DEG ELEVATING HANGER .....                       | 129 |
| 90 DEG ELEVATING CALF PAD .....                     | 131 |
| FOOTPLATES.....                                     | 131 |
| KIDS ANGLE ADJ FOOTPLATE FRONT MOUNT .....          | 132 |
| ADULT ANGLE-ADJ FOOTPLATE FRONT MOUNT.....          | 134 |
| ALRS .....                                          | 135 |
| EXT MOUNT ALRS.....                                 | 135 |
| EXT MOUNT ALR HANGERS.....                          | 135 |
| ARTICULATING HANGER .....                           | 136 |
| ARTICULATING LEGREST DISC. 12/12/02.....            | 138 |
| ARTICULATING HANGER CALF PAD .....                  | 140 |
| EXTENSION TUBES.....                                | 140 |
| ROUND EXTENSION TUBE .....                          | 141 |
| FOOTPLATES.....                                     | 142 |
| COMPOSITE FOOTPLATE.....                            | 143 |
| ADULT ANGLE-ADJUSTABLE FOOTPLATE EXT MOUNT.....     | 145 |
| KIDS ANGLE ADJ FOOTPLATE ASSM-EXTENSION MOUNT ..... | 147 |
| FRONT MOUNT ALRS.....                               | 148 |
| 90 DEG ARTICULATING HANGERS.....                    | 148 |
| 90 DEG ARTICULATING HANGER .....                    | 149 |
| 90 DEG ARTICULATING HANGER CALF PAD .....           | 151 |
| FOOTPLATES.....                                     | 151 |
| KIDS ANGLE ADJ FOOTPLATE FRONT MOUNT .....          | 152 |
| ADULT ANGLE-ADJ FOOTPLATE FRONT MOUNT.....          | 154 |

|                                                 |     |
|-------------------------------------------------|-----|
| MULTI-POSITION LATCH STUD SUPPORT .....         | 156 |
| ARMRESTS .....                                  | 156 |
| ARMRESTS.....                                   | 156 |
| LENGTH ADJUSTABLE LOCKING FLIP-UP ARMREST ..... | 157 |
| T POST ARMREST.....                             | 159 |
| FLIP-UP ARMREST .....                           | 161 |
| ADJUSTABLE LOCKING FLIP-UP ARMREST .....        | 162 |
| SINGLE POST HEIGHT ADJUSTABLE ARMREST .....     | 164 |
| ARMREST RECEIVERS .....                         | 165 |
| KIDS SINGLE POST ARMREST RECEIVER .....         | 166 |
| ADULT SINGLE POST ARMREST RECEIVER.....         | 167 |
| ACCESSORIES .....                               | 168 |
| POSITIONING BELTS .....                         | 168 |
| AUTO BUCKLE POSITIONING BELTS .....             | 169 |
| AIRCRAFT BUCKLE POSITIONING BELT.....           | 170 |
| POSITIONING BELT ATTACHMENT KIT.....            | 171 |
| PADDED DOUBLE PULL HIP BELT.....                | 172 |
| TRAYS .....                                     | 172 |
| TUFFAK TRAY (MANUAL & PEDIATRIC CHAIRS) .....   | 173 |
| TRAY HARDWARE.....                              | 175 |
| TRAY RIBS TUFFAK.....                           | 176 |
| TOUCH-UP PAINT.....                             | 177 |
| PLASTIC SIDE GUARD .....                        | 178 |
| IMPACT GUARD NEOPRENE & GEL VELCRO PADS.....    | 179 |
| BAGS AND POUCHES .....                          | 179 |
| BACKPACKS AND SEAT POUCHES.....                 | 180 |
| CASE LOGIC BAGS.....                            | 181 |
| CELL PHONE PACK.....                            | 182 |
| ONE-ARM DRIVE RETRO KIT .....                   | 183 |
| SOFT FEEL LEG STRAP .....                       | 185 |
| ADJUSTABLE LEG STRAP .....                      | 186 |
| SPOKE GUARDS.....                               | 187 |
| TOOL KIT .....                                  | 188 |
| TIRE PUMP .....                                 | 189 |

**NOTE:** THESE ITEMS USED FOR CHAIRS WITH 17"-18" SEAT DEPTH. (19"-20" FRAME DEPTH).

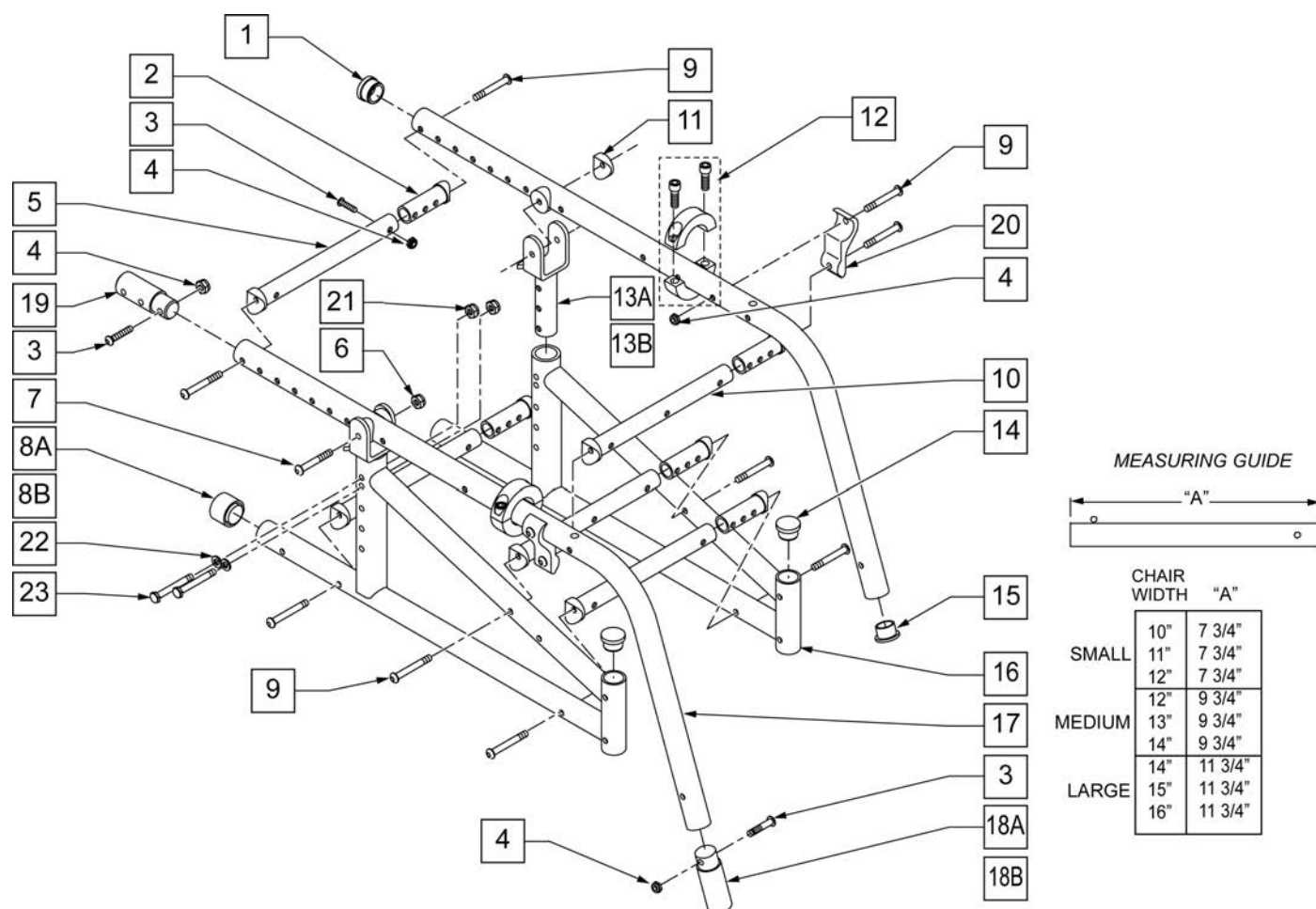

| Pos. | Item Number | Description                 | Remarks      |
|------|-------------|-----------------------------|--------------|
| 1    | 483039      | 1" INSERT PLUG ALUM ZP TS   |              |
| 2    | 483184      | STRUT SADDLE GROWING        |              |
| 3    | 106961      | 1/4-28 x 1 1/4 BH C/S       |              |
| 4    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS |              |
| 5    | 522802      | STRUT TUBE 10"ZP/KZ         | 10 -12" WIDE |
| 5    | 522804      | STRUT TUBE 12" ZP/P5/KZ     | 12 -14" WIDE |
| 5    | 522806      | STRUT TUBE 14" ZP/P5/KZ     | 14 -16" WIDE |
| 6    | 106934      | 1/4-28 NYLOCK NUT           |              |
| 7    | 106964      | 1/4-28 x 2 BH C/S           |              |
| 8A   | 561014      | 1" MED CAP                  |              |
| 8B   | 561018      | 1" TUBE END BUSHING THICK   |              |
| 9    | 226130      | 1/4-28 x 1 1/2 BH C/S       |              |
| 10   | 522829      | STRUT TUBE 10" ZPTS GRW     | 10 -12" WIDE |
| 10   | 522830      | STRUT TUBE 12" ZPTS GRW     | 12 -14" WIDE |
| 10   | 522831      | STRUT TUBE 14" ZPTS GRW     | 14 -16" WIDE |
| 11   | 561004      | 1" SADDLE 1/4" HOLE         |              |
| 12   | 342048      | ADJ SEAT STOP 1"            |              |

| Pos.                 | Item Number | Description                      | Remarks                   |
|----------------------|-------------|----------------------------------|---------------------------|
| 13A                  | 505251      | TILT BRACKET ZP TS 45°           | 45° TILT                  |
| 13B*                 | 505250      | TILT BRACKET ZP TS 30°           | 30° TILT                  |
| 14                   | 561005      | 1" INSERT PLUG .125              |                           |
| 15                   | 561002      | 1" INSERT PLUG .083              |                           |
| 8,14,16              | 660903S     | S/F PNTD ZP TS 1 1/8"            | (SPECIFY COLOR)           |
| 2-4,9                | 920461      | STRUT TUBE HDWR KT ZP TS GRW     |                           |
| 1,15,17              | 660900S     | TILT SEAT TUBE PNTD ZP TS        | (SPECIFY COLOR)           |
| 18A                  | 483052      | FOOTREST EXT 3" ZP TS            |                           |
| 18B                  | 483053      | FOOTREST EXT 5" ZP TS            |                           |
| 19*                  | 483040      | REAR FRAME EXT .083 (15"-16") ZP | SEE NOTE ABOVE.           |
| 20                   | 550119      | STRUT TUBE MOUNT BRACKET         |                           |
| 21                   | 106935      | NUT NYLOCK THIN 1/4-28 ROHS      |                           |
| 22                   | 106944      | 1/4" AN WASHER FLAT THIN         |                           |
| 23                   | 106971      | 14-28 x 1 3/8 HEX C/S            |                           |
| 3,4,18               | 920316      | TS 3" FOOTREST EXT W/HDWR        |                           |
| 3,4,18B              | 920317      | TS 5" FOOTREST EXT W/HDWR        |                           |
| 3,4,19*              | 920318      | ZP REAR FRAME EXT 2" W/HDWR      | SEE NOTE ABOVE.           |
| 5,9,10               | 675080      | TS STRT TBE KIT GROWING SMALL    | DOES NOT INCLUDE SADDLES. |
| 5,9,10               | 675081      | TS STRT TBE KIT GROWING MED      |                           |
| 5,9,10               | 675082      | TS STRT TBE KIT GROWING LARGE    |                           |
| 2-4,6-8,11,13A,14,16 | 920301S     | ZP TS SIDE FRAME W/HDWR          |                           |


**NOTE:** When growing chair to 15" or 16". an additional cylinder lock mechanism is needed and item 8B is used with 12" rear wheel.

**NOTE:** For locking mechanism, see Section-16, *Mechanical Lock and Strut Assemblies*.

**NOTE:** Part 505250 (TILT BRACKET ZP TS 30° ) Discontinued (Parts Only) and is only used on chairs prior to 9/21/02.

\*\* These items used for chairs with (15" -16") seat depth (17" -19" Frame Depth).

\*\*\* For serial numbers prior to ZP-56988, in the 11", 13", 15" and 16" sizes, please order part number 483184 (STRUT SADDLE GROWING) QTY of 2 per tube.

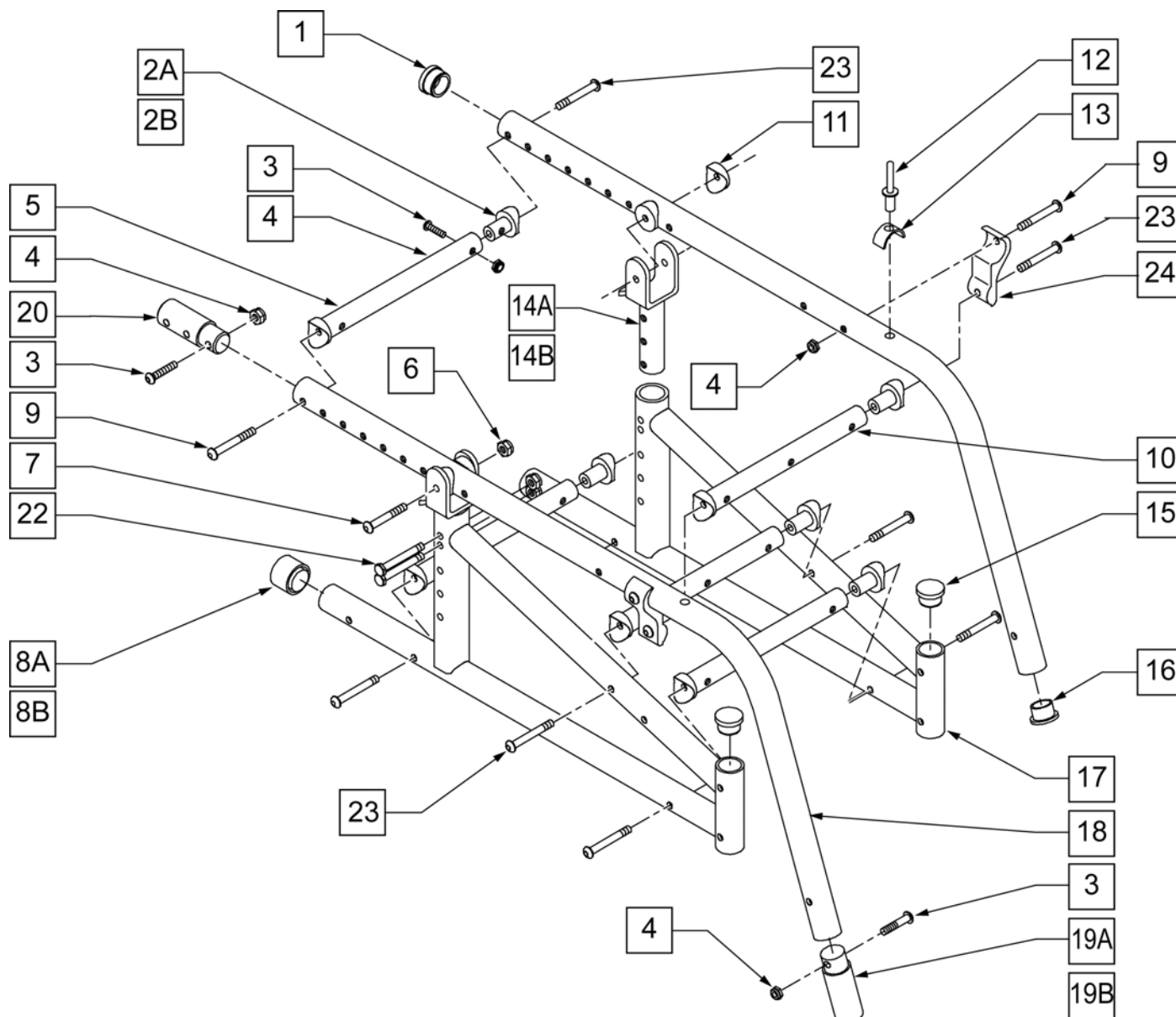


| Pos. | Item Number | Description                 | Remarks      |
|------|-------------|-----------------------------|--------------|
| 1    | 483039      | 1" INSERT PLUG ALUM ZP TS   |              |
| 2A   | 483184      | STRUT SADDLE ZGS            |              |
| 2B   | 483161      | STRUT SADDLE S525           |              |
| 3    | 106961      | 1/4-28 x 1 1/4 BH C/S       |              |
| 4    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS |              |
| 5**  | 522802      | STRUT TUBE 10" ZP/KZ        | 10 -12" WIDE |



| Pos.          | Item Number | Description                           | Remarks       |
|---------------|-------------|---------------------------------------|---------------|
| 5**           | 522804      | STRUT TUBE 12" ZP/P5/KZ               | 12 -14" WIDE  |
| 5**           | 522806      | STRUT TUBE 14" ZP/P5/KZ               | 14 -16" WIDE  |
| 6             | 106934      | 1/4-28 NYLOCK NUT                     |               |
| 7             | 106964      | 1/4-28 X 1 3/8 BH C/S                 |               |
| 8A            | 561014      | 1" MED CAP                            |               |
| 8B            | 561018      | 1" TUBE END BUSHING THICK             |               |
| 9             | 226130      | 1/4-28 x 1 1/2 BH C/S                 |               |
| 10***         | 522829      | STRUT TUBE 10" ZPTS GRW               | 10 -12" WIDE  |
| 10***         | 522830      | STRUT TUBE 12" ZPTS GRW               | 12 -14" WIDE  |
| 10***         | 522831      | STRUT TUBE 14" ZPTS GRW               | 14 -16" WIDE  |
| 11            | 561004      | 1" SADDLE 1/4" HOLE                   |               |
| 12            | 240108      | RIVET POP SSD64SSBS                   |               |
| 13            | 550489      | SEAT STOP                             |               |
| 14A           | 505251      | TILT BRACKET ZP TS 45°                |               |
| 14B*          | 505250      | TILT BRACKET ZP TS 30°                |               |
| 15            | 561005      | 1" INSERT PLUG .125                   |               |
| 16            | 561002      | 1" INSERT PLUG .083                   |               |
| 8,15,17       | 660903S     | S/F PNTD ZP TS 1 1/8" (SPECIFY COLOR) |               |
| 2B,3,4,9      | 920344      | STRUT TUBE HARDWARE KIT ZP TS         |               |
| 1,12,13,16,18 | 660900S     | TILT SEAT TUBE PNTD ZP TS             | SPECIFY COLOR |
| 19A           | 483052      | FOOTREST EXT 3" ZP TS                 |               |
| 19B           | 483053      | FOOTREST EXT 5" ZP TS                 |               |
| 20            | 483040      | REAR FRAME EXT .083 (15"-16") ZP      |               |
| 21            | 550119      | STRUT TUBE MOUNT BRACKET              |               |
| 22            | 106971      | 1/4-28 x 1 3/8 HEX C/S                |               |
| 3,4,19A       | 920316      | TS 3" FOOTREST EXT W/ HARDWARE        |               |
| 3,4,19B       | 920317      | TS 5" FOOTREST EXT W/ HARDWARE        |               |
| 3,4,20        | 920318      | ZP REAR FRAME EXT 2" W/ HARDWARE      |               |
| 2A-5,9,10     | 675096      | STRUT TUBE RTRO GRWTH KT TS -SMALL    |               |
| 2A-5,9,10     | 675097      | STRUT TUBE RTRO GRWTH KT TS -MED      |               |
| 2A-5,9,10     | 675098      | STRUT TUBE RTRO GRWTH KT TS -LARGE    |               |
| 4,9,14A,21    | 901730      | 45° TS RETRO KIT                      |               |
| 23            | 226267      | 1/4-28 x 1 5/8 BH C/S W/PATCH         |               |

**NOTE:** \* THESE ITEMS USED FOR CHAIRS WITH 17"-18" SEAT DEPTH. (19"-20" FRAME DEPTH).

\*\* 505250 (TILT BRACKET ZP TS 30) DISCONTINUED (PARTS ONLY), AND IS USED ON CHAIRS PRIOR TO 9/21/02.

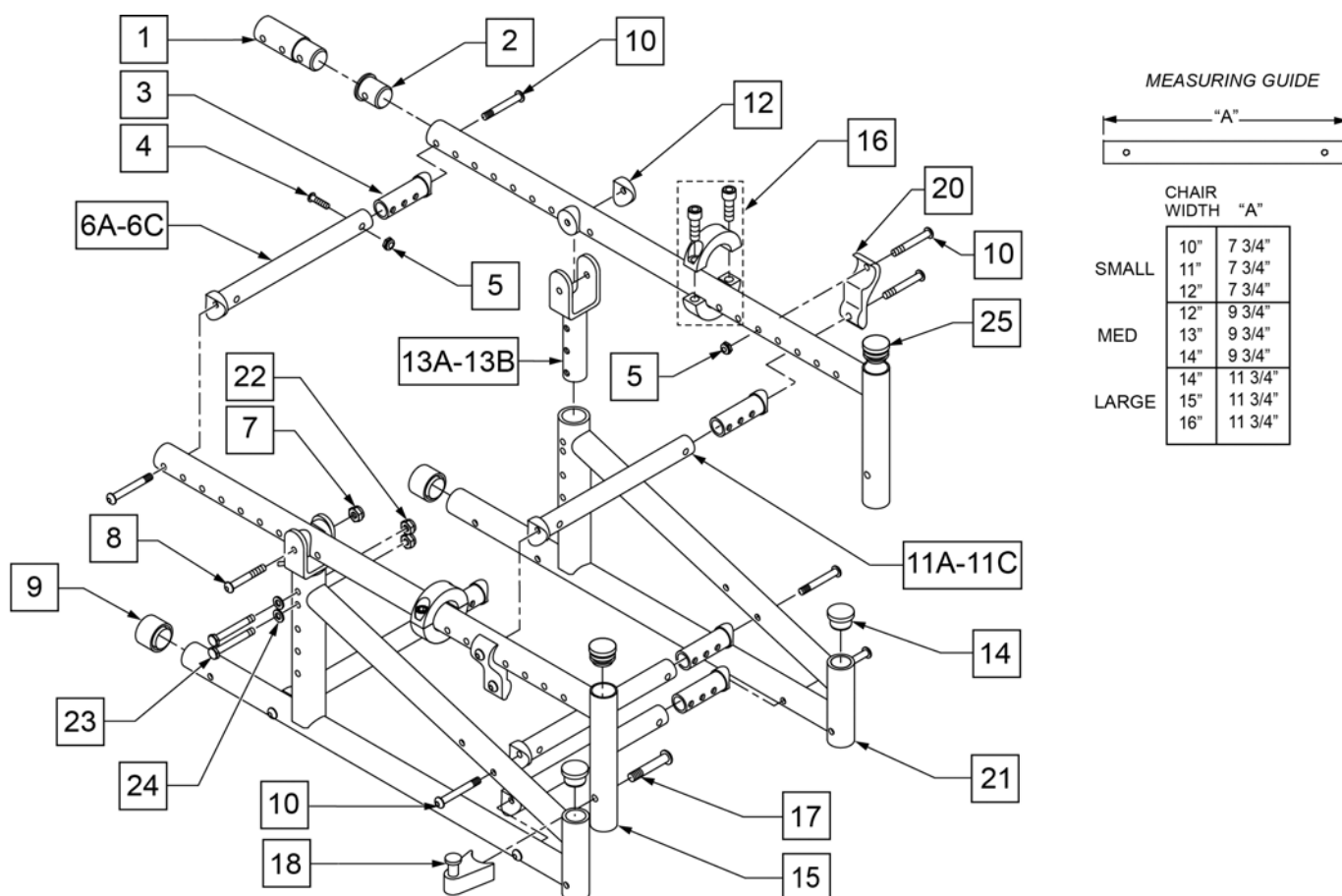

| Pos.  | Item Number | Description                    | Remarks      |
|-------|-------------|--------------------------------|--------------|
| 1*    | 483088      | REAR FRAME EXT .065 15"-16" ZP | SEE NOTE:    |
| 2     | 483089      | SEAT TUBE PLUG S/A TS/P5       |              |
| 3     | 483184      | STRUT SADDLE ZGS               |              |
| 4     | 106961      | 1/4-28 x 1 1/4 BH C/S          |              |
| 5     | 106935      | NUT NYLOCK THIN 1/4-28 ROHS    |              |
| 6A    | 522802      | STRUT TUBE 10" ZP/KZ           | 10 -12" WIDE |
| 6B    | 522804      | STRUT TUBE 12" ZP/P5/KZ        | 12 -14" WIDE |
| 6C    | 522806      | STRUT TUBE 14" ZP/P5/KZ        | 14 -16" WIDE |
| 7     | 106934      | 1/4-28 NYLOCK NUT              |              |
| 8     | 106964      | 1/4-28 x 2 BH C/S              |              |
| 9     | 561014      | 1" MED CAP                     |              |
| 10    | 226130      | 1/4-28 X 1 1/2 HEX C/S         |              |
| 11A   | 522829      | STRUT TUBE TS 10" ZP TS        | 10 -12" WIDE |
| 11B   | 522830      | STRUT TUBE TS 12" ZP TS        | 12 -14" WIDE |
| 11C   | 522831      | STRUT TUBE TS 14" ZP TS        | 14 -16" WIDE |
| 12    | 561004      | 1" SADDLE 1/4" HOLE            |              |
| 13A   | 505251      | TILT BRACKET ZP TS 45°         |              |
| 13B** | 505250      | TILT BRACKET ZP TS 30°         | SEE NOTE:    |
| 14    | 561005      | 1" INSERT PLUG .125            |              |

| Pos.              | Item Number | Description                    | Remarks                   |
|-------------------|-------------|--------------------------------|---------------------------|
| 16                | 342048      | ADJ SEAT STOP 1"               |                           |
| 17                | 106982      | BHCS 5/16-18 X 1-3/4 SHLD ROHS |                           |
| 20                | 550119      | STRUT TUBE MOUNT BRACKET       |                           |
| 22                | 106935      | NUT NYLOCK THIN 1/4-28 ROHS    |                           |
| 23                | 106964      | 1/4-28 x 2 BH C/S              |                           |
| 24                | 106944      | 1/4 AN WASHER FLAT THIN        |                           |
| 25                | 560735      | 7/8" ROUND TUBE PLUG           | Use in place of SA Hanger |
| 1,4,5*            | 920309      | TS S/A 2" SEAT EXT W/HDWR      | SEE NOTE:                 |
| 1,4,5,15A         | 920908S     | SEAT TUBE S/A TS 19-20(NEW PR) | SPECIFY COLOR.            |
| 2,4,5,15B         | 660909S     | SEAT TUBE S/A TS (NEW)         | SPECIFY COLOR.            |
| 3-5,10            | 920461      | STRUT TUBE HDWR KT ZP TS GRW   |                           |
| 17, 18            | 107491      | LATCH STUD KIT                 |                           |
| 6A,10,11A         | 675080      | TS STRT TBE KIT GROWING SMALL  | SADDLES NOT INCLUDED.     |
| 6B,10,11B         | 675081      | TS STRT TBE KIT GROWING MED    | SADDLES NOT INCLUDED.     |
| 6C,10,11C         | 675082      | TS STRT TBE KIT GROWING LARGE  | SADDLES NOT INCLUDED.     |
| 5,10,13A,20       | 901730      | 45° TS RETRO KIT               |                           |
| 5,7-9,12-14,22-24 | 920301S     | ZP TS SIDE FRAME W/ HDWR       |                           |


**NOTE:** When growing chair to 15" or 16". an additional cylinder lock mechanism is needed and item 8B is used with 12" rear wheel.

**NOTE:** For locking mechanism, see Section-16, *Mechanical Lock and Strut Assemblies*.

**NOTE:** Part 505250 (TILT BRACKET ZP TS 30° ) Discontinued (Parts Only) and is only used on chairs prior to 9/21/02.

\*\* These items used for chairs with (15" -16") seat depth (17" -19" Frame Depth).


\*\*\* For serial numbers prior to ZP-56988, in the 11", 13", 15" and 16" sizes, please order part number 483184 (STRUT SADDLE GROWING) QTY of 2 per tube.

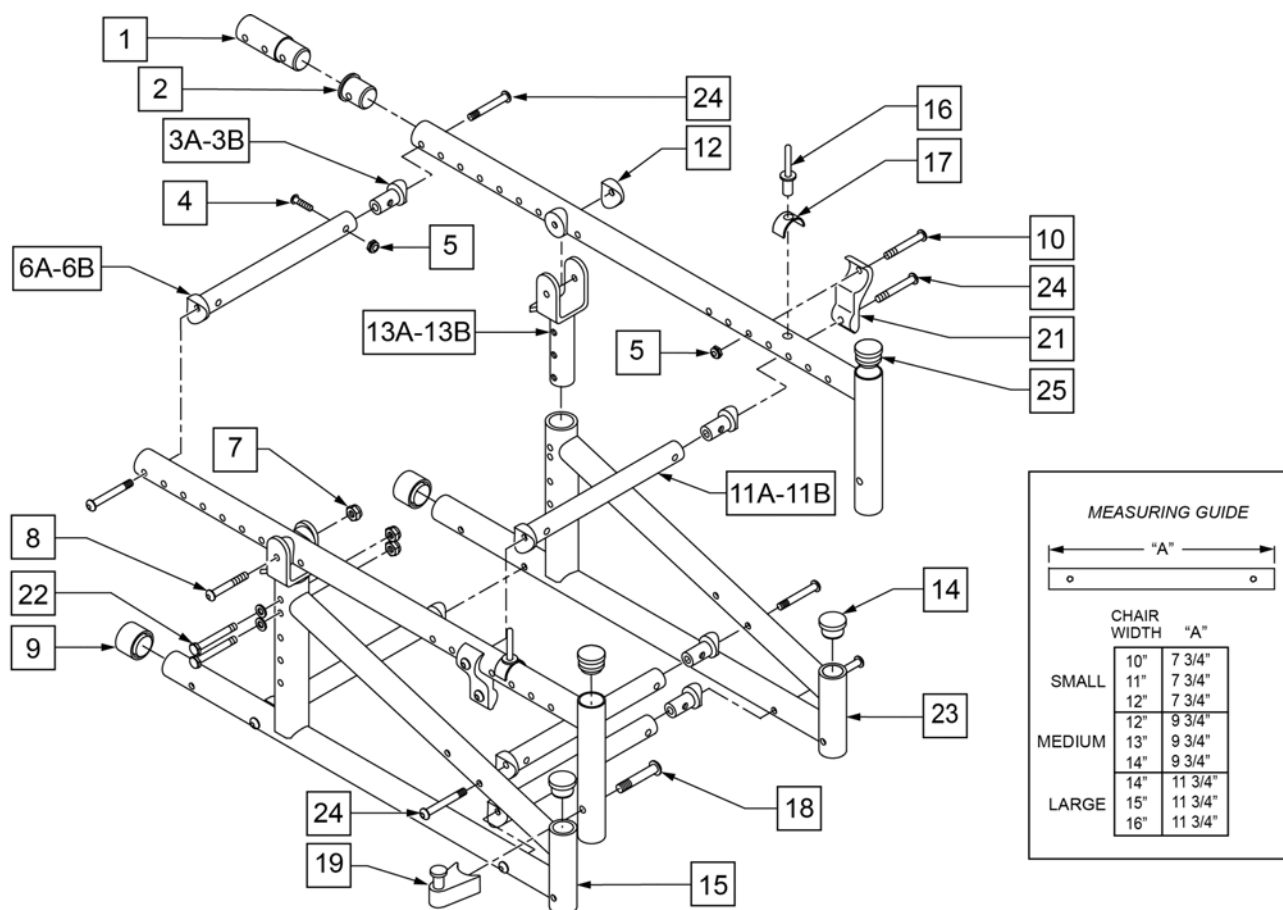

| Pos.  | Item Number | Description                    | Remarks      |
|-------|-------------|--------------------------------|--------------|
| 1**   | 483088      | REAR FRAME EXT .065 15"-16" ZP | SEE NOTE:    |
| 2     | 483089      | SEAT TUBE PLUG S/A TS/P5       |              |
| 3A    | 483184      | STRUT SADDLE ZGS               |              |
| 3B    | 483161      | STRUT SADDLE S525              |              |
| 4     | 106961      | 1/4-28 x 1 1/4 BH C/S          |              |
| 5     | 106935      | NUT NYLOCK THIN 1/4-28 ROHS    |              |
| 6A*** | 522802      | STRUT TUBE ZP/KZ 10"           | 10 -12" WIDE |
| 6B*** | 522804      | STRUT TUBE 12" ZP/P5/KZ        | 12 -14" WIDE |
| 6C*** | 522806      | STRUT TUBE 14" ZP/P5/KZ        | 14 -16" WIDE |

| Pos.              | Item Number | Description                        | Remarks      |
|-------------------|-------------|------------------------------------|--------------|
| 7                 | 106934      | 1/4-28 NYLOCK NUT                  |              |
| 8                 | 106964      | 1/4-28 x 2 BH C/S                  |              |
| 9                 | 561014      | 1" MED CAP                         |              |
| 10                | 226130      | 1/4-28 x 1 1/2 BH C/S              |              |
| 11A***            | 522829      | STRUT TUBE TS 10" ZP TS            | 10 -12" WIDE |
| 11B***            | 522830      | STRUT TUBE TS 12" ZP TS            | 12 -14" WIDE |
| 11C***            | 522831      | STRUT TUBE TS 14" ZP TS            | 14 -16" WIDE |
| 12                | 561004      | 1" SADDLE 1/4" HOLE                |              |
| 13A*              | 505251      | TILT BRACKET ZP TS 45°             |              |
| 13B*              | 505250      | TILT BRACKET ZP TS 30°             | SEE NOTE:    |
| 14                | 561005      | 1" INSERT PLUG .125                |              |
| 16                | 240108      | RIVET POP SSD64SSBS                |              |
| 17                | 550489      | SEAT STOP                          |              |
| 18                | 106982      | BHCS 5/16-18 X 1-3/4 SHLD ROHS     |              |
| 21                | 550119      | STRUT TUBE MOUNT BRACKET           |              |
| 22                | 106971      | 1/4-28 x 1 3/8 HEX C/S             |              |
| 1,4,5**           | 920309      | TS S/A 2" SEAT EXT W/HDWR          | SEE NOTE:    |
| 2,15-17           | 660910S     | SEAT TUBE S/A PNTD P5              |              |
| 3B,4,5,10         | 920344      | STRUT TUBE HDWR KIT ZP TS          |              |
| 18, 19            | 107491      | LATCH STUD KIT                     |              |
| 3A-6A,10,11A      | 675096      | STRUT TUBE RETRO GRWTH KT TS SMALL |              |
| 3A-6B,10,11B      | 675097      | STRUT TUBE RETRO GRWTH KT TS MED   |              |
| 3A-6C,10,11C      | 675098      | STRUT TUBE RETRO GRWTH KT TS LARGE |              |
| 13A,5,10,21       | 901730      | 45° TS RETRO KIT                   |              |
| 5,7,8,12-14,22,23 | 920301S     | ZP TS SIDE FRAME W/HDWR            |              |
| 24                | 226267      | 1/4-28 x 1 5/8 BH C/S W/PATCH      |              |

**NOTE:** Effective on Zippie TS after S/N ZP-56988.

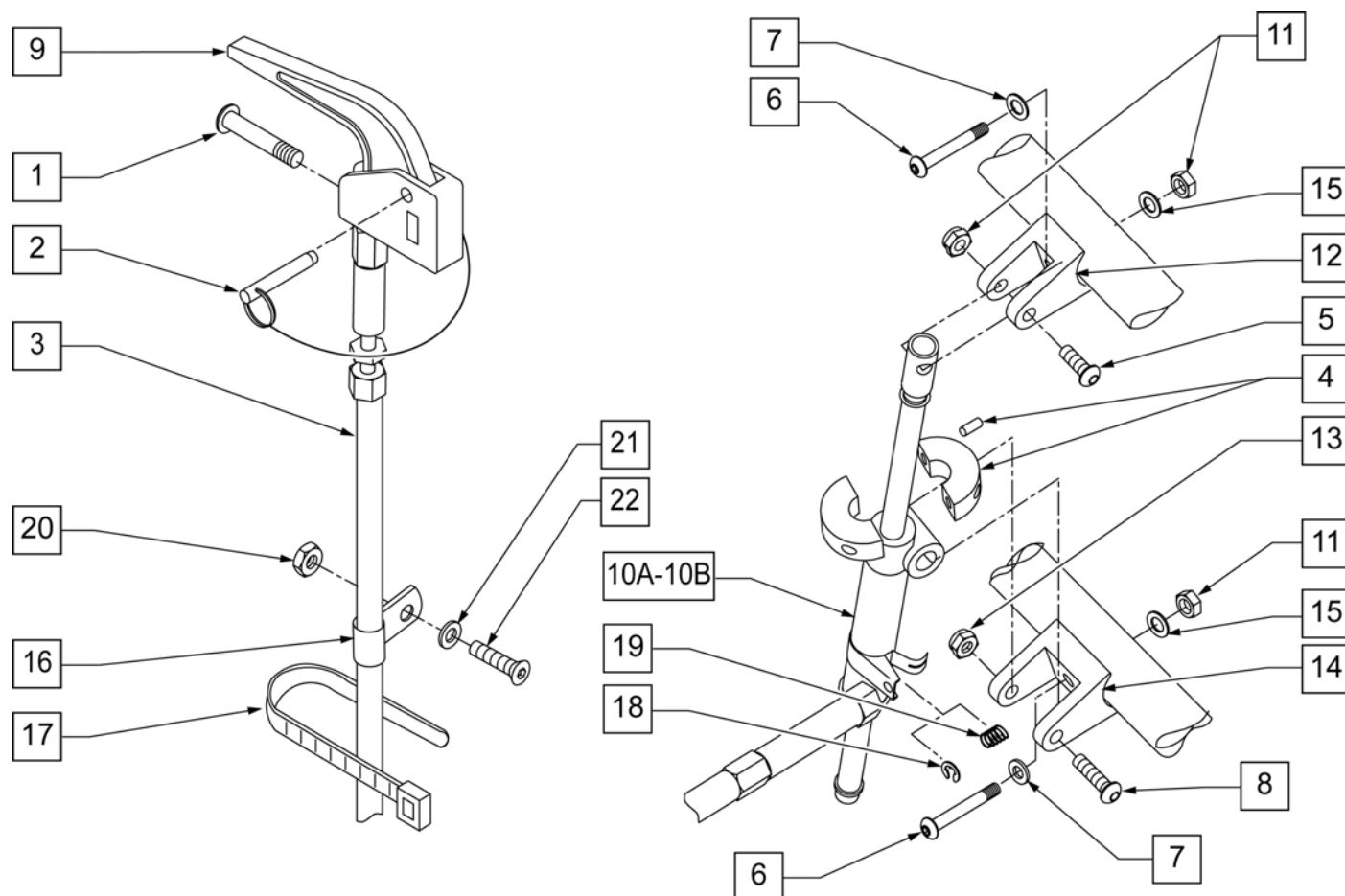


| Pos. | Item Number | Description                      | Remarks |
|------|-------------|----------------------------------|---------|
| 1    | 281120      | 10-32 x 5/8 PAN HD PHIL          |         |
| 2    | 349021      | TRIGGER RELEASE PIN TS/QT/P5     |         |
| 3    | 342057      | CYLINDER LOCK CABLE ASSEMBLY     |         |
| 4    | 342049      | 10mm ID SPLIT COLLAR             |         |
| 5    | 106961      | 1/4-28 x 1 1/4 BH C/S            |         |
| 6    | 226130      | 1/4-28 x 1 1/2 BH C/S            |         |
| 7    | 106943      | 1/4 AN WASHER FLAT               |         |
| 8    | 106982      | BHCS 5/16-18 X 1-3/4 SHLD ROHS   |         |
| 9    | 580110      | CYL LOCK LEVER ASSEMBLY QT/QR/P5 |         |
| 10A  | 120625      | MECH CYLINDER LOCK R BR RECLINER |         |
| 10B  | 120626      | MECH CYLINDER LOCK L BR RECLINER |         |
| 11   | 106935      | NUT NYLOCK THIN 1/4-28 ROHS      |         |
| 12   | 500140      | CYLINDER BRACKET 1" TS/P5        |         |
| 13   | 106937      | 5/16-18 NYLOCK NUT THIN          |         |
| 14   | 500141      | CYLINDER BRACKET TS 1 1/2"       |         |
| 15   | 262130      | 6mm WASHER WAVE                  |         |
| 16   | 349511      | 1/2" ID PLASTIC CLAMP BLK        |         |
| 17   | 560410      | ZIP TIE BLACK                    |         |

| Pos.              | Item Number | Description                     | Remarks |
|-------------------|-------------|---------------------------------|---------|
| 18                | 342000      | 1/4" E-CLIP                     |         |
| 19                | 321141      | .650L x 6 COIL SPRING           |         |
| 20                | 200100-QU   | 10-32 NYLOCK NUT                |         |
| 21                | 261204      | #10 AN WASHER .063              |         |
| 22                | 226032      | 10-32 x 1 1/2 BH C/S BLK        |         |
| 1-3,5-9,10A,11-22 | 920493      | MECH CYL LOCK ASSM R TS 45° NEW | RIGHT   |
| 1-3,5-9,10B,11-22 | 920494      | MECH CYL LOCK ASSM L TS 45° NEW | LEFT    |

**NOTE:** Discontinued 11/16/99 fits Zippie TS prior to S/N ZP-56988.

**NOTE:** 45 DEG MECHANICAL CYLINDER LOCK ASSEMBLY IS STANDARD BEGINNING 11/13/93.


\* ITEM 4 REQUIRED ON ALL MECH LOCK ASSEMBLIES FOR CHAIRS WITH THE FOLLOWING CONFIGURATIONS: FIXED FRONT 15" AND 16"; 10" SEAT DEPTH; S/A MODELS WITH 6" AND 8" CASTERS. AND 2" EXTENSIONS; FIXED MODELS WITH 8" CASTER AND 2" EXTENSION.

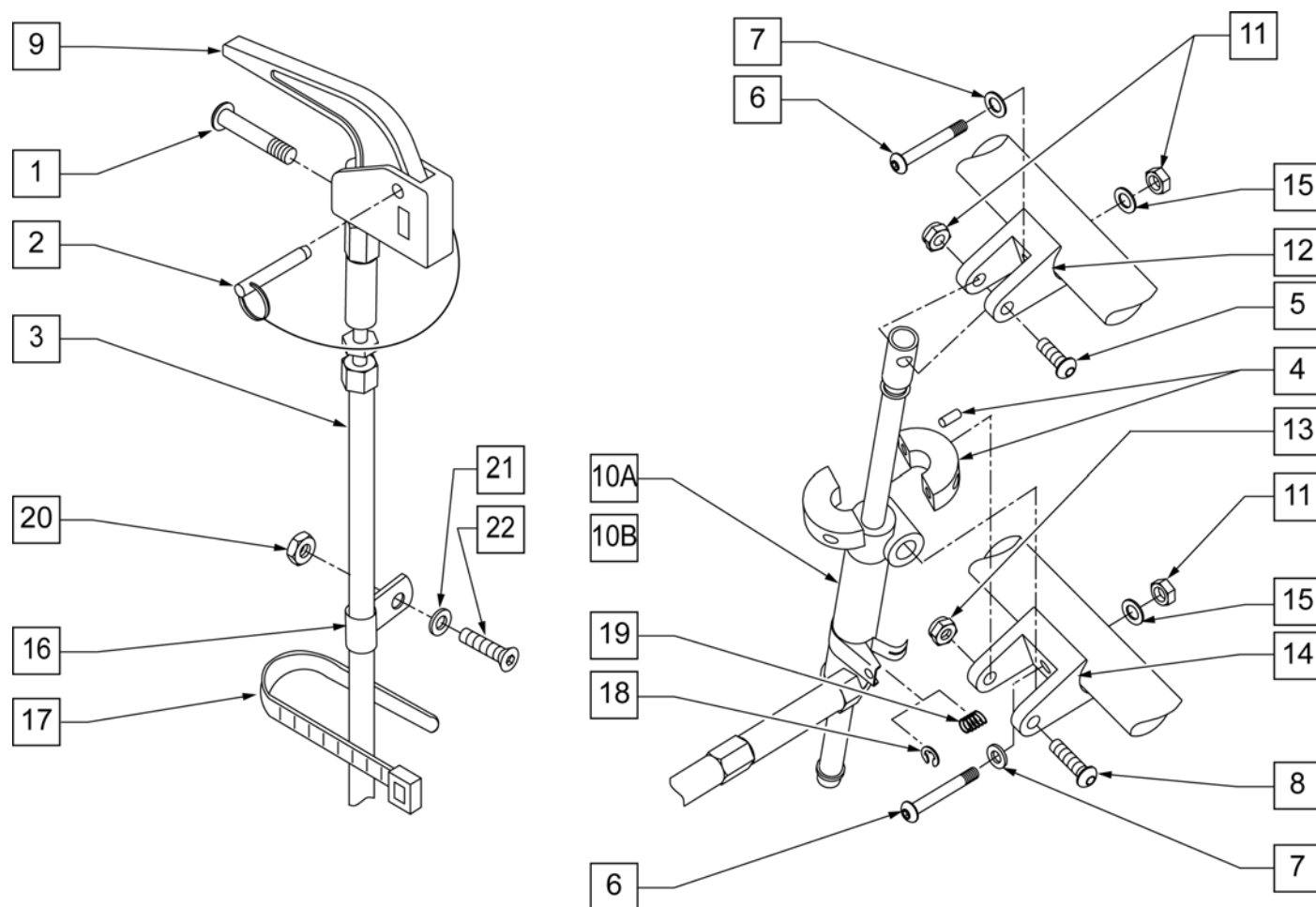

| Pos. | Item Number | Description                      | Remarks   |
|------|-------------|----------------------------------|-----------|
| 1    | 281120      | 10-32 x 5/8 PAN HD PHIL          |           |
| 2    | 349021      | TRIGGER RELEASE PIN TS/QT/P5     |           |
| 3    | 342057      | CYLINDER LOCK CABLE ASSEMBLY     |           |
| 4*   | 342049      | 10mm ID SPLIT COLLAR             |           |
| 5    | 106961      | 1/4-28 x 1 1/4 BH C/S            |           |
| 6    | 226130      | 1/4-28 x 1 1/2 BH C/S            |           |
| 7    | 106943      | 1/4 AN WASHER FLAT               |           |
| 8    | 106982      | BHCS 5/16-18 X 1-3/4 SHLD ROHS   |           |
| 9    | 580110      | CYL LOCK LEVER ASSEMBLY QT/QR/P5 |           |
| 10A  | 120625      | MECH CYLINDER LOCK R BR RECLINER | 45° RIGHT |
| 10A  | 120626      | MECH CYLINDER LOCK L BR RECLINER | 45° LEFT  |
| 10B  | 342051      | MECH CYLINDER LOCK R             | 30° RIGHT |
| 10B  | 342052      | MECH CYLINDER LOCK L             | 30° LEFT  |



| Pos.              | Item Number | Description                     | Remarks |
|-------------------|-------------|---------------------------------|---------|
| 11                | 106935      | NUT NYLOCK THIN 1/4-28 ROHS     |         |
| 12                | 500140      | CYLINDER BRACKET 1" TS/P5       |         |
| 13                | 106937      | 5/16-18 NYLOCK NUT THIN         |         |
| 14                | 500141      | CYLINDER BRACKET TS 1 1/2"      |         |
| 15                | 262130      | 6mm WASHER WAVE                 |         |
| 16                | 349511      | 1/2" ID PLASTIC CLAMP BLK       |         |
| 17                | 560410      | ZIP TIE BLACK                   |         |
| 18                | 342000      | 1/4" E-CLIP                     |         |
| 19                | 321141      | .650L x 6 COIL SPRING           |         |
| 20                | 200100-QU   | 10-32 NYLOCK NUT                |         |
| 21                | 261204      | #10 AN WASHER .063              |         |
| 22                | 226032      | 10-32 x 1 1/2 BH C/S BLK        |         |
| 1-3,5-9,10A,11-22 | 920415      | MECH CYL LOCK ASSEMBLY 45° R TS | RIGHT   |
| 1-3,5-9,10A,11-22 | 920405      | MECH CYL LOCK ASSEMBLY 45° L TS | LEFT    |
| 1-10B,11-22       | 920315      | MECH CYL LOCK ASSEMBLY 30° R TS | RIGHT   |
| 1-10B,11-22       | 920305      | MECH CYL LOCK ASSEMBLY 30° L TS | LEFT    |

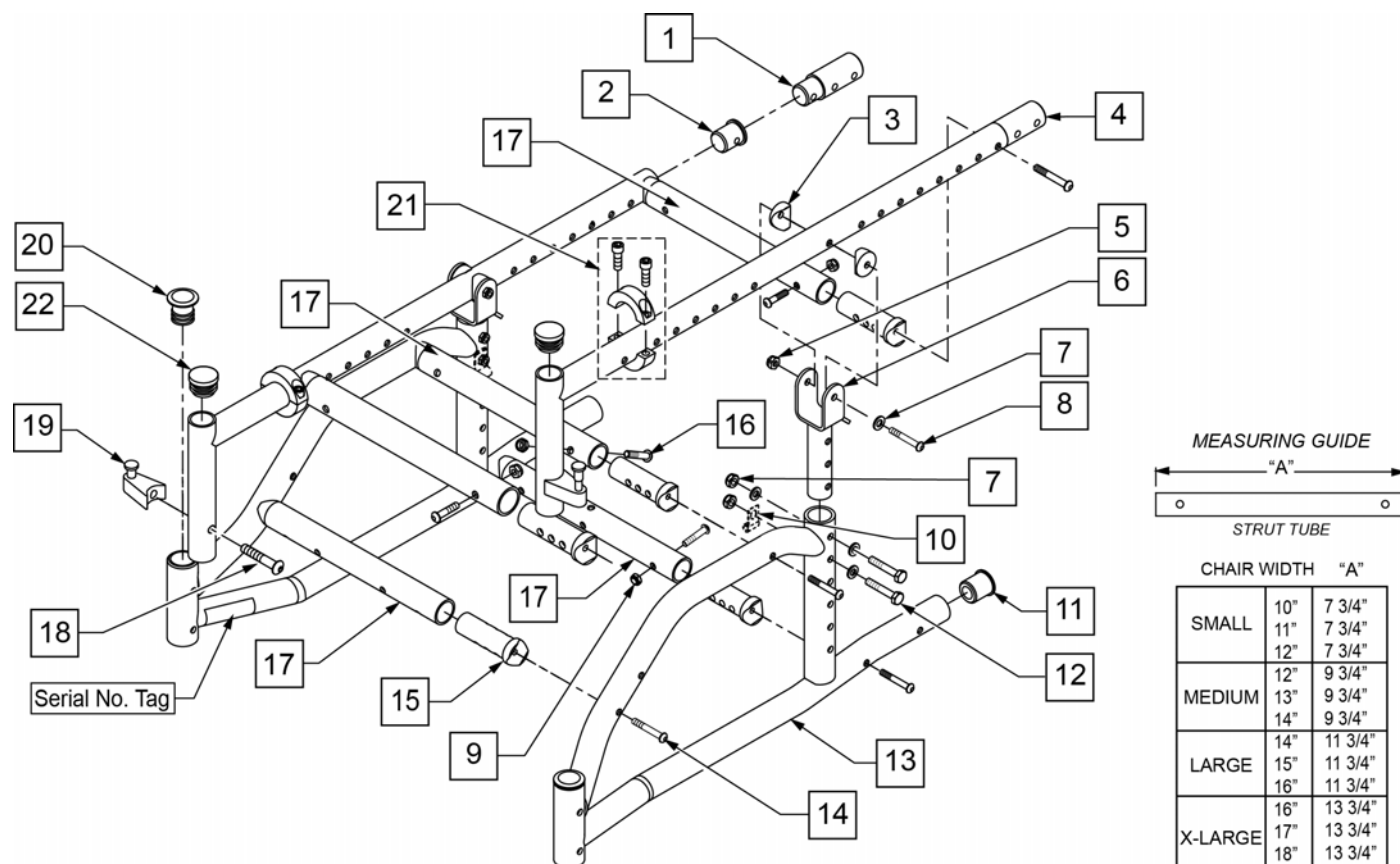


| Pos.         | Item Number | Description                     | Remarks                                |
|--------------|-------------|---------------------------------|----------------------------------------|
| 1            | 483088      | REAR FRAME EXT .065 15 -16" ZP  |                                        |
| 2            | 483089      | SEAT TUBE PLUG S/A TS/P5        |                                        |
| 3            | 561004      | 1" SADDLE 1/4" HOLE             |                                        |
| 5            | 106934      | NUT NYLOCK 1/4 -28 ROHS         |                                        |
| 6            | 505251      | TILT BRACKET ZP TS 45°          |                                        |
| 7            | 106943      | WASER FLAT AN 1/4 ROHS          |                                        |
| 8            | 106965      | BHCS 1/4 -28 x 2 SHLDR ROHS     |                                        |
| 9            | 106935      | NUT NYLOCK THIN 1/4 -28 ROHS    |                                        |
| 10           | 112388      | P-CLIP 3/16" VINYL COAT         |                                        |
| 11           | 561018      | 1" TUBE END BUSHING THICK       |                                        |
| 12           | 106972      | HHCS 1/4 -28 x 1 1/2 ROHS       |                                        |
| 14           | 107028      | BHCS 1/4 -28 x 1 1/2 ROHS       |                                        |
| 15           | 483184      | STRUT SADDLE ZGS                |                                        |
| 16           | 106961      | BHCS 1/4 -28 x 1 1/4 ROHS       |                                        |
| 18           | 106982      | BHCS 5/16 -18 x 1 3/4 SHLD ROHS |                                        |
| 19           | 105787      | S/A LATCH STUD SUPPORT NUT      |                                        |
| 20           | 561062      | 1" INSERT PLUG .065 RIBS        |                                        |
| 21           | 342048      | ADJ SEAT STOP 1"                |                                        |
| 22           | 560735      | 7/8" ROUND TUBE PLUG            | Use in place of SA Hanger              |
| 1,14         | 920309      | TS S/A 2" SEAT EXT W/HDWR       |                                        |
| 1,4,16,18,19 | 115713S     | SEAT TUBE TS 19-20D W/ LTCH     | WORKS W/ 19-20 SEAT DEPTHS             |
| 2,4,16,18,19 | 115714S     | SEAT TUBE TS 13-18D W/ LTCH     | WORKS W/ 13-18 SEAT DEPTHS             |
| 9,14-17      | 115717      | STRUT TUBE SML ASSM             | SMALL - WORKS W/ 10" -12" SEAT WIDTHS  |
| 9,14-17      | 115718      | STRUT TUBE MED ASSM             | MEDIUM - WORKS W/ 12" -14" SEAT WIDTHS |
| 9,14-17      | 115719      | STRUT TUBE LRG ASSM             | LARGE - WORKS W/ 14" -16" SEAT WIDTHS  |

| Pos.     | Item Number | Description                  | Remarks                                 |
|----------|-------------|------------------------------|-----------------------------------------|
| 9,14-17  | 115720      | STRUT TUBE X-L ASSM          | X-LARGE - WORKS W/ 16' -18" SEAT WIDTHS |
| 11,13,20 | 115715S     | RIGID ZTS S/F ASSM           | RIGHT                                   |
| 11,13,20 | 115716S     | RIGID ZTS S/F ASSM           | LEFT                                    |
| 14,15,16 | 920461      | STRUT TUBE HDWR KT ZP TS GRW |                                         |
| 18,19    | 107491      | LATCH STUD KIT               |                                         |


Note: When replacing a Zippie TS SE Side Frame, a New Zippie TS Side Frame must be purchased. The additional functionality of a third width adjustment hole is the only difference.

Note: Prior to serial number Z1A-034388 & Z1SE-008579, please replace X-hinges in pairs.

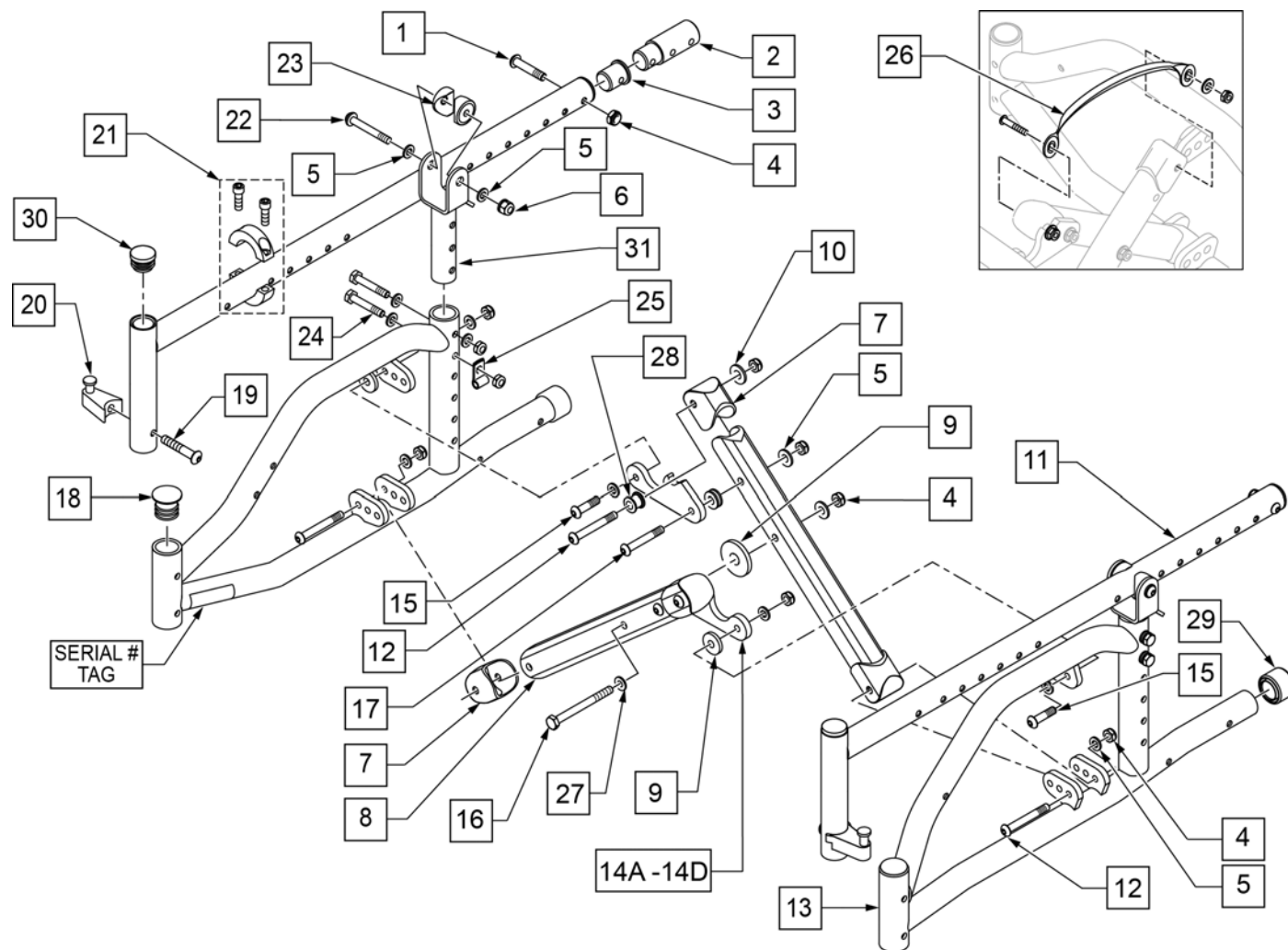

| Pos. | Item Number | Description                    | Remarks                            |
|------|-------------|--------------------------------|------------------------------------|
| 1    | 106961      | BHCS 1/4 -28 x 1 1/4 ROHS      |                                    |
| 2    | 483088      | REAR FRAME EXT .065 15 -16" ZP |                                    |
| 3    | 483089      | SEAT TUBE PLUG S/A TS/P5       |                                    |
| 4    | 106935      | NUT NYLOCK THIN 1/4 -28 ROHS   |                                    |
| 5    | 106943      | WASHER FLAT AN 1/4 ROHS        |                                    |
| 6    | 106934      | NUT NYLOCK 1/4 -28 ROHS        |                                    |
| 7    | 562845      | END CAP OVAL TUBE ZGS          |                                    |
| 9    | 561016      | 1" WASHER 5/16 HOLE            |                                    |
| 10   | 106945      | WASHER FLAT SAE 1/4 ROHS       |                                    |
| 12   | 106964      | BHCS 1/4 -28 x 2 ROHS          |                                    |
| 14A  | 120996      | X-HINGE 10-12IN ZGS            | SMALL - WORKS W/ 10-12 SEAT WIDTHS |


| Pos.                        | Item Number | Description                     | Remarks                              |
|-----------------------------|-------------|---------------------------------|--------------------------------------|
| 14B                         | 120997      | X-HINGE 12-14IN ZGS             | MEDIUM - WORKS W/ 12-14 SEAT WIDTHS  |
| 14C                         | 120998      | X-HINGE 14-16IN ZGS             | LARGE - WORKS W/ 14-16 SEAT WIDTHS   |
| 14D                         | 120999      | X-HINGE 16-18IN ZGS             | X-LARGE - WORKS W/ 16-18 SEAT WIDTHS |
| 15                          | 107355      | BHCS 1/4 -28 x 1" ROHS          |                                      |
| 16                          | 109265      | HHCS 1/4 -28 x 2 1/2 ROHS       |                                      |
| 17                          | 106962      | BHCS 1/4 -28 x 1 3/4 ROHS       |                                      |
| 18                          | 561062      | 1" INSERT PLUG .065 RIBS        |                                      |
| 19                          | 106982      | BHCS 5/16 -18 x 1 3/4 SHLD ROHS |                                      |
| 21                          | 342048      | ADJ SEAT STOP 1"                |                                      |
| 22                          | 106965      | BHCS 1/4 -28 x 2 SHLDR ROHS     |                                      |
| 23                          | 561004      | 1" SADDLE 1/4" HOLE             |                                      |
| 24                          | 106972      | HHCS 1/4 -28 x 1 1/2 ROHS       |                                      |
| 25                          | 112388      | P-CLIP 3/16" VINYL COAT         |                                      |
| 26                          | 750552      | ZGS FOLDING STRAP               |                                      |
| 27                          | 561028      | .25 NYLON WASHER                |                                      |
| 28                          | 481089      | Z2 X-HINGE SPACER               |                                      |
| 29                          | 561018      | 1" TUBE END BUSHING THICK       |                                      |
| 30                          | 560735      | 7/8" ROUND TUBE PLUG            | Use in place of SA Hanger            |
| 31                          | 505251      | TILT BRACKET                    |                                      |
| 1,2,4,11,19,20              | 115713S     | SEAT TUBE TS 19-20D W/ LTCH     | WORKS W/ 19-20 SEAT DEPTHS           |
| 1,3,4,11,19,20              | 115714S     | SEAT TUBE TS 13-18D W/ LTCH     | WORKS W/ 13-18 SEAT DEPTHS           |
| 13,18,29                    | 115721S     | FOLDING ZTS S/F ASSM R          | RIGHT                                |
| 13,18,29                    | 115722S     | FOLDING ZTS S/F ASSM L          | LEFT                                 |
| 19,20                       | 107491      | LATCH STUD KIT                  |                                      |
| 4-5,7-10,12,14A,15-17,27,28 | 673100      | X-TUBE ASSM 10-12" ZGS          | SMALL - WORKS W/ 10-12 SEAT WIDTHS   |
| 4-5,7-10,12,14B,15-17,27,28 | 673101      | X-TUBE ASSM 12-14" ZGS          | MEDIUM - WORKS W/ 12-14 SEAT WIDTHS  |
| 4-5,7-10,12,14B,15-17,27,28 | 673102      | X-TUBE ASSM 14-16" ZGS          | LARGE - WORKS W/ 14-16 SEAT WIDTHS   |
| 4-5,7-10,12,14B,15-17,27,28 | 673103      | X-TUBE ASSM 16-18" ZGS          | X-LARGE - WORKS W/ 16-18 SEAT WIDTHS |

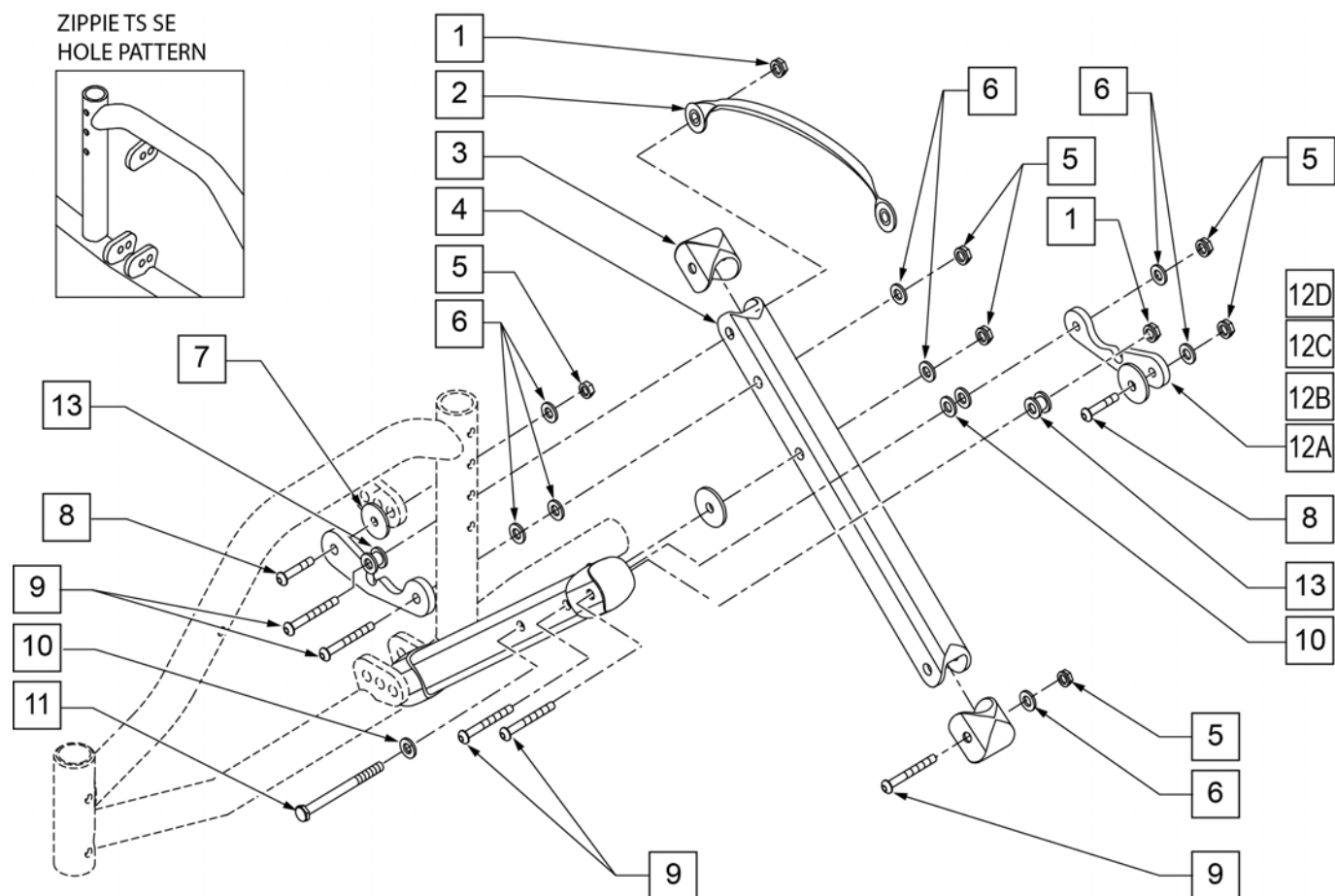

| Pos.        | Item Number | Description                  | Remarks                              |
|-------------|-------------|------------------------------|--------------------------------------|
| 1           | 106934      | 1/4-28 NYLOCK NUT            |                                      |
| 2           | 750552      | ZGS FOLDING STRAP            |                                      |
| 3           | 562845      | END CAP OVAL TUBE ZGS        |                                      |
| 5           | 106935      | NUT NYLOCK THIN 1/4-28 ROHS  |                                      |
| 6           | 106943      | 1/4 AN WASHER FLAT           |                                      |
| 7           | 561016      | 1" WASHER 5/16" HOLE         |                                      |
| 8           | 226111      | 1/4-28 x 1 BH C/S SHLDR      |                                      |
| 9           | 106964      | 1/4-28 x 2 BH C/S            |                                      |
| 10          | 561028      | 1/4" NYLON WASHER            |                                      |
| 11          | 221181      | 1/4-28 x 2 1/2 G8 HEX C/S    |                                      |
| 12A         | 120996      | X-HINGE 10-12IN ZGS          | SMALL - WORKS W/ 10-12 SEAT WIDTHS   |
| 12B         | 120997      | X-HINGE 12-14IN ZGS          | MEDIUM - WORKS W/ 12-14 SEAT WIDTHS  |
| 12C         | 120998      | X-HINGE 14-16IN ZGS          | LARGE - WORKS W/ 14-16 SEAT WIDTHS   |
| 12D         | 120999      | X-HINGE 16-18IN ZGS          | X-LARGE - WORKS W/ 16-18 SEAT WIDTHS |
| 13          | 481089      | Z2 X-HINGE SPACER            |                                      |
| 14          | 226140      | 1/4-28 x 1 3/4 BH C/S        |                                      |
| 15          | 106945      | 1/4" SAE WASHER              |                                      |
| 1-12A,13-15 | 673100      | X TUBE ASSEMBLY 10"- 12" ZGS | 10 -12" WIDE                         |
| 1-12B,13-15 | 673101      | X TUBE ASSEMBLY 12"- 14" ZGS | 12 -14" WIDE                         |

| Pos.        | Item Number | Description                  | Remarks      |
|-------------|-------------|------------------------------|--------------|
| 1-12C,13-15 | 673102      | X TUBE ASSEMBLY 14"- 16" ZGS | 14 -16" WIDE |
| 1-12D,13-15 | 673103      | X TUBE ASSEMBLY 16"- 18" ZGS | 16 -18" WIDE |

**NOTE:** \* ITEM 4A USED WITH SMALL (10-12") AND MEDIUM (12-14") FRAME WIDTHS. ITEM 4B USED WITH LARGE (14-16") AND X-LARGE (16-18") FRAME WIDTHS.

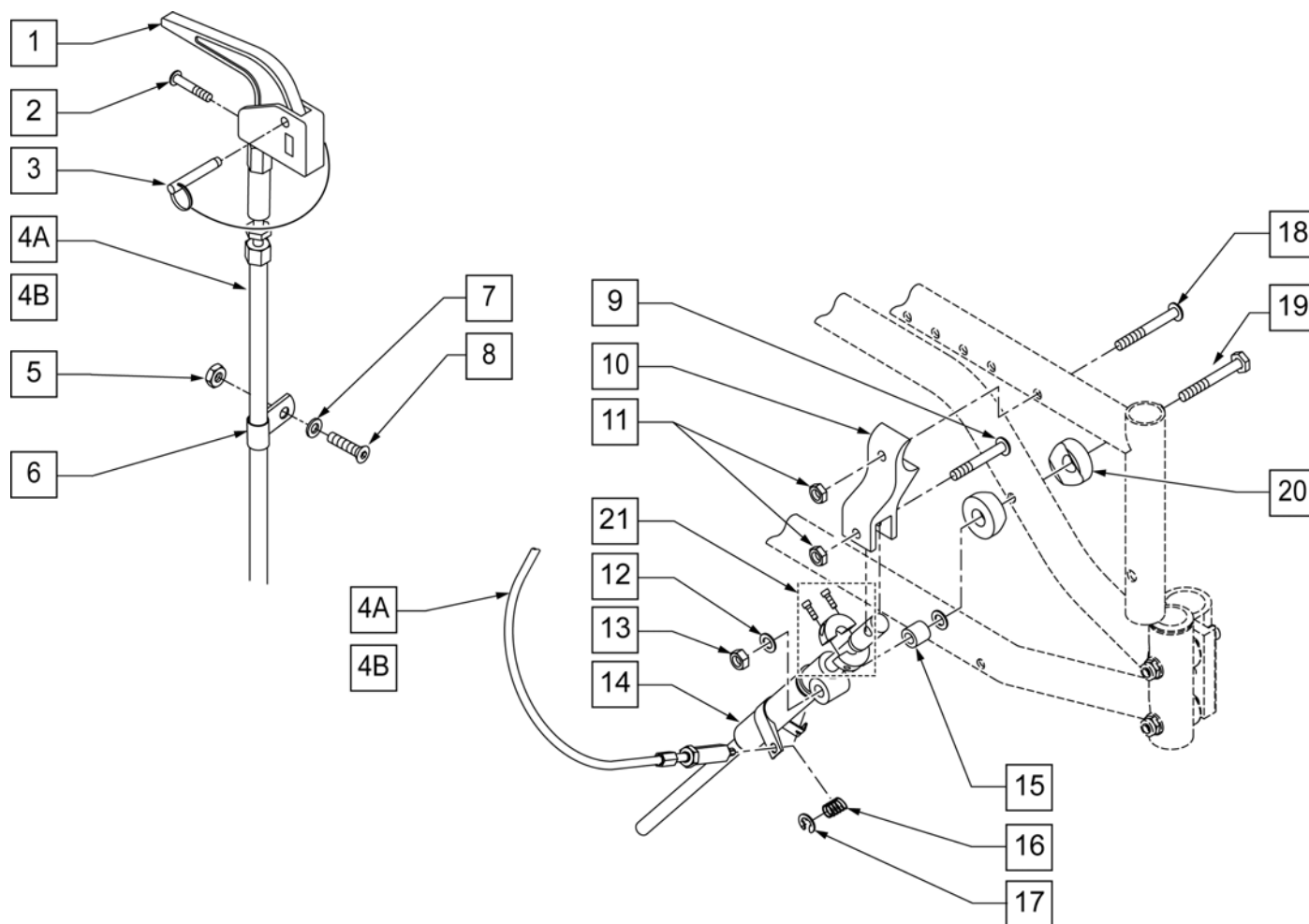


| Pos. | Item Number | Description                  | Remarks |
|------|-------------|------------------------------|---------|
| 1    | 580110      | CYL LOCK LEVER ASSM QT/QR/P5 |         |
| 2    | 281120      | 10/32 x 5/8 PAN HD PHIL      |         |
| 3    | 349021      | TRIGGER REL PIN TS/QT/P5     |         |
| 4A*  | 342057      | CYL LOCK CABLE ASSM          |         |
| 4B*  | 342056      | CYL LOCK CABLE ASSM T45      |         |
| 5    | 200100-QU   | 10-32 NYLOCK NUT             |         |
| 6    | 349511      | 1/2" ID PLST CLAMP BLK       |         |
| 7    | 261204      | #10 AN WASHER .063           |         |
| 8    | 226032-QU   | 10-32 x 1 1/2 BH C/S BLK     |         |
| 9    | 226131      | 1/4-28 x 1 1/4 BH C/S SHLDR  |         |
| 10   | 500609      | FLDG ZTS MCHLCK MNT SEAT FRM |         |
| 11   | 106935      | NUT NYLOCK THIN 1/4-28 ROHS  |         |
| 12   | 106943      | 1/4 AN WASHER FLAT           |         |
| 13   | 106934      | 1/4-28 NYLOCK NUT            |         |
| 14   | 120625      | MECH CYLINDER LOCK BR RCLNR  | RIGHT   |
| 14   | 120626      | MECH CYLINDER LOCK BR RCLNR  | LEFT    |





| Pos.            | Item Number | Description                      | Remarks |
|-----------------|-------------|----------------------------------|---------|
| 15              | 520916      | FLDG ZTS MCHLCK PVT SPCR FRM     |         |
| 16              | 321141      | .650L x 6 COIL SPRING            |         |
| 17              | 342000      | 1/4" E-CLIP                      |         |
| 18              | 226130      | 1/4-28 x 1 1/2 BH C/S            |         |
| 19              | 221193      | 1/4-28 x 2 3/4 G8 HEX C/S        |         |
| 20              | 561004      | 1" SADDLE 1/4" HOLE              |         |
| 21              | 342049      | 10mm ID SPLIT COLLAR             |         |
| 1-3,4A,5-21     | 920652      | MECH LOCK ASSM R SHT CBL NEW TS  | RIGHT   |
| 1-3,4A,5-21     | 920653      | MECH LOCK ASSM L SHT CBL NEW TS  | LEFT    |
| 1-3,4B,5-21     | 920771      | MECH LOCKI ASSM R LNG CBL NEW TS | RIGHT   |
| 1-3,4B,5-21     | 920772      | MECH LOCKI ASSM L LNG CBL NEW TS | LEFT    |
| 11,13,14,17     | 922055      | ZTS MECH LOCK REPAIR KIT         |         |
| 1-4A,6,14-17,21 | 922240      | LOCKING MECH KIT SML/MED ZPTS    | RIGHT   |
| 1-4A,6,14-17,21 | 922241      | LOCKING MECH KIT SML/MED ZPTS    | LEFT    |
| 1-4B,6,14-17,21 | 922242      | LOCKING MECH KIT LRG/XLG ZPTS    | RIGHT   |
| 1-4B,6,14-17,21 | 922243      | LOCKING MECH KIT LRG/XLG ZPTS    | LEFT    |


NOTE: Fits Zippie TS chairs with s/n prefix ZP, ZT, ZTS, ZTE, Z1A, & Z1SE.

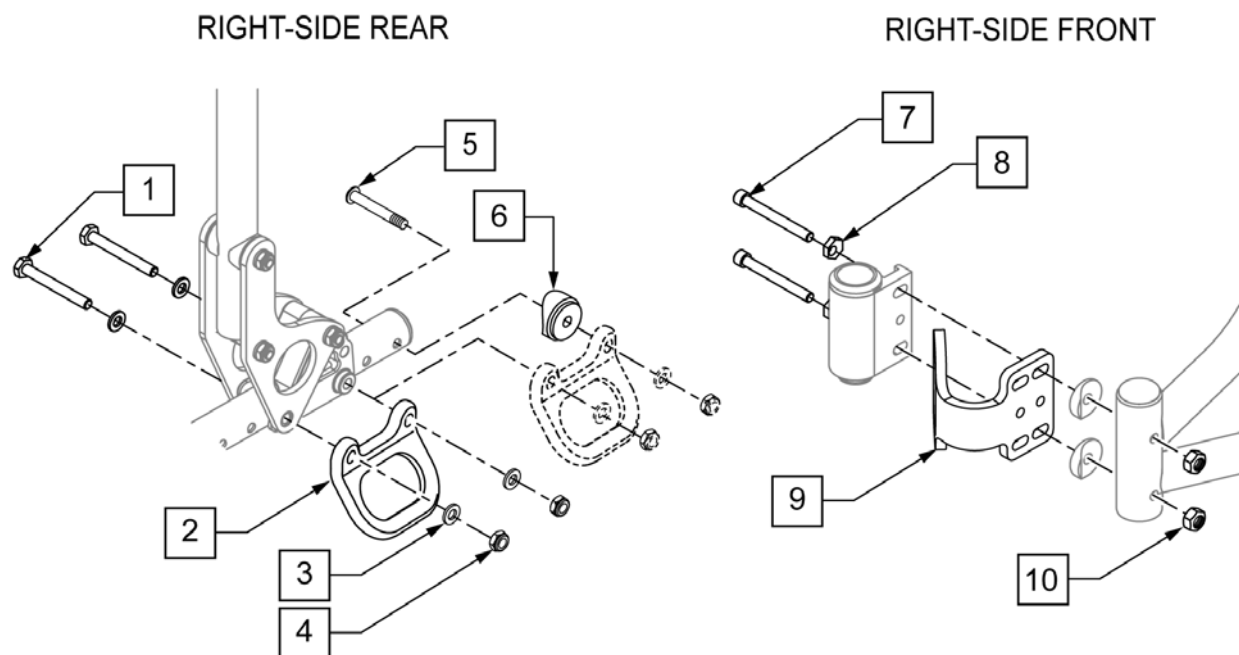


| Pos. | Item Number | Description                  |
|------|-------------|------------------------------|
| 1-9  | 922970S     | TRANSIT KIT ZIPPIE           |
| 1    | 107351      | HHCS 1/4-28 X 2 1/4 ROHS     |
| 3    | 106943      | WASHER FLAT AN 1/4 ROHS      |
| 4    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS  |
| 5    | 226150      | 1/4-28 X 2 BH C/S            |
| 6    | 560518      | SADDLE BACKREST Q2 DAB RIGHT |
| 6    | 560519      | SADDLE BACKREST Q2 DAB LEFT  |
| 7    | 221202      | 1/4-28 X 2-1/8" C/S CLR ZN   |
| 8    | 481430      | ECCENTRIC WASHER TRANSPORT   |
| 10   | 201120      | 1/4-28 AUTOMATION NUT        |
| 1-10 | 105169      | TRANSIT KIT ZIPPIE TS        |
| 1-10 | 105175      | TRANSIT KIT NEW ZIPPIE TS    |

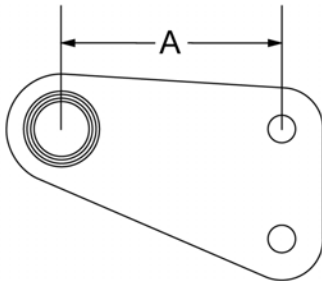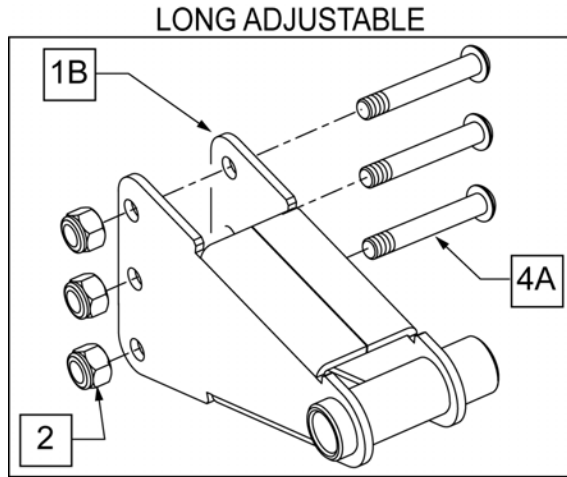









| AXLE  | DIM "A" |
|-------|---------|
| SHORT | 2.0"    |
| LONG  | 4.0"    |

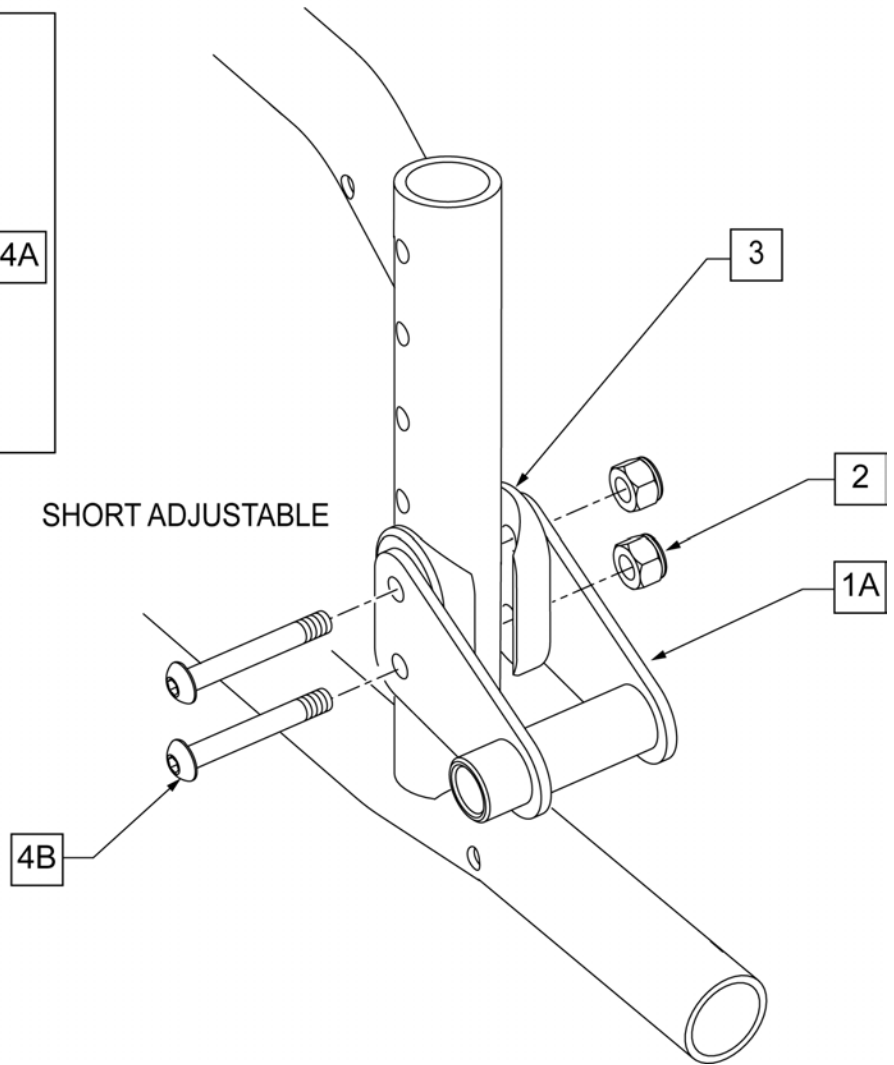


| Pos.  | Item Number | Description                   | Remarks                      |
|-------|-------------|-------------------------------|------------------------------|
| 1A    | 505320      | QR AXLE PLATE ZP TS R         | SHORT-RIGHT                  |
| 1A    | 505321      | QR AXLE PLATE ZP TS L         | SHORT-LEFT                   |
| 1B    | 506120      | AXLE BRACKET RIGHT            | LONG-RIGHT                   |
| 1B    | 506121      | AXLE BRACKET LEFT             | LONG-LEFT                    |
| 2     | 106934      | NUT NYLOCK 1/4 -28 ROHS       |                              |
| 3     | 561085      | 1 1/8" HALF SADDLE 2 HOLE QTS |                              |
| 4A    | 106962      | BHCS 1/4 -28 x 1 3/4 ROHS     | USE W/ LONG AXLE PLATE       |
| 4B    | 106964      | BHCS 1/4 -28 x 2 ROHS         | USE W/ SHORT AXLE PLATE      |
| 1A-4B | 115726      | AXLE BRACKET SHT R KIT        | RIGHT - SHORT AXLE PLATE KIT |
| 1A-4B | 115727      | AXLE BRACKET SHT L KIT        | LEFT - SHORT AXLE PLATE KIT  |
| 1B-4A | 115728      | AXLE BRACKET LNG R KIT        | RIGHT - LONG AXLE PLATE KIT  |
| 1B-4A | 115729      | AXLE BRACKET LNG L KIT        | LEFT - LONG AXLE PLATE KIT   |

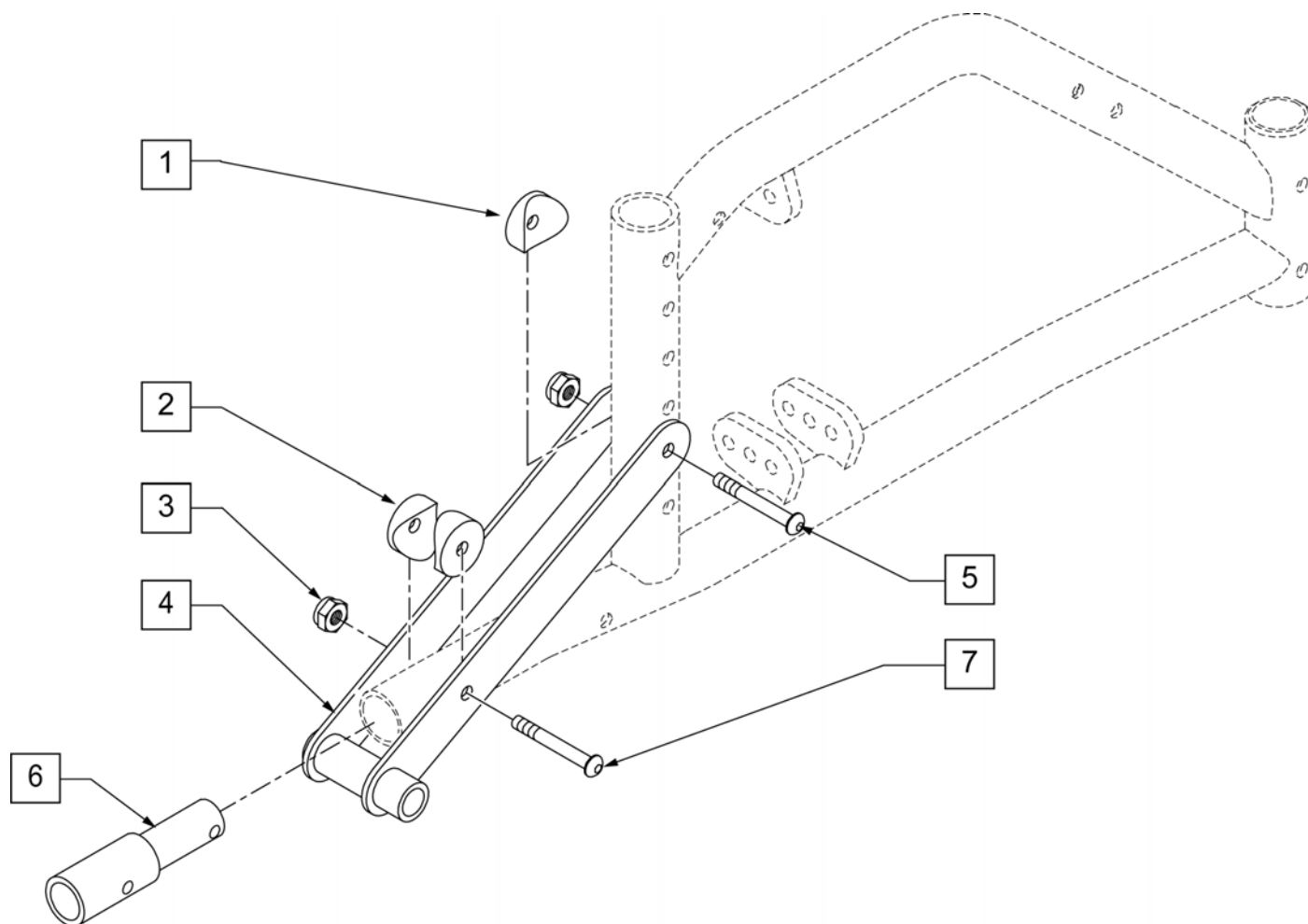


| Pos. | Item Number | Description                          |
|------|-------------|--------------------------------------|
| 1    | 561001      | 1 1/8" SADDLE 1/4" HOLE              |
| 2    | 561004      | 1" SADDLE 1/4" HOLE                  |
| 3    | 106935      | NYLOCK NUT THIN 1/4 -28 ROHS         |
| 5    | 106962      | BHCS 1/4-28 x 1 3/4 ROHS             |
| 6    | 109734      | TUBE EXTENSION FRAME                 |
| 7    | 112390      | BHCS 1/4-28 X 1-3/4 FULL ROHS        |
| 1-7  | 920161      | QR AXLE PLATE TS 12" WHEEL W/ AT REC |
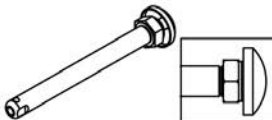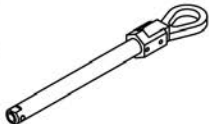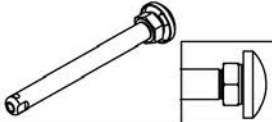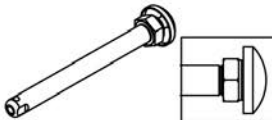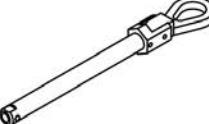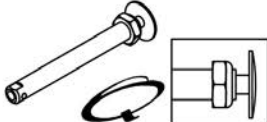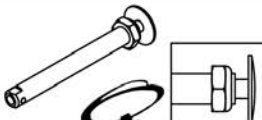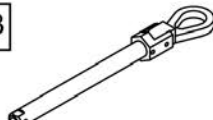
| Wheel Type                              | Axle Type          |                                                                                         | Axle P/N           | Spacer P/N                                                                                    |
|-----------------------------------------|--------------------|-----------------------------------------------------------------------------------------|--------------------|-----------------------------------------------------------------------------------------------|
| Mag Wheels<br>12" Axle Plate            | Quick Release      | 1A    | 102065<br>A=4 3/4  | N/A                                                                                           |
|                                         | Quad Quick Release | 1B     | 116390<br>A=4 3/4  | N/A                                                                                           |
| Mag Wheels<br>STD & LONG<br>Axle Plates | Quick Release      | 1E    | 102063<br>A= 4 1/4 | N/A                                                                                           |
|                                         | Quick Release      | 1A    | 102065<br>A= 4 3/4 | 2A  920117 |
|                                         | Quad Quick Release | 1B     | 116390<br>A= 4 3/4 | 2A  920117 |
| Lite<br>Spoke Wheels<br>(20 -24")       | Quick Release      | 1C   | 107268<br>A=4 1/4  | N/A                                                                                           |
| 18" Spoke                               | Quick Release      | 1D  | 102064<br>A=4 1/2  | N/A                                                                                           |
|                                         | Quad Quick Release | 1B   | 922170<br>A=4 1/2  | N/A                                                                                           |


Quick Release Measuring Guide

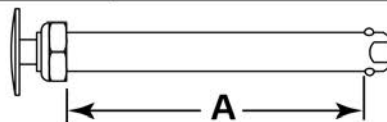


| Pos. | Item Number | Description                      |
|------|-------------|----------------------------------|
| 1A   | 102065      | QR AXLE MAG W/M-12 NUT           |
| 1B   | 116390      | QR AXLE W/ QUAD NUT PUSH PUTTON  |
| 1C   | 107268      | QR AXLE AND NUT SHORT BIG BUTTON |
| 1D   | 102064      | QR AXLE M12 NUT CLR BIG BUTTON   |
| 1E   | 102063      | QR AXLE MAG W/M-12 NUT           |
| 2A   | 920117      | MAG SPACER 6S W/ SET SCREW       |

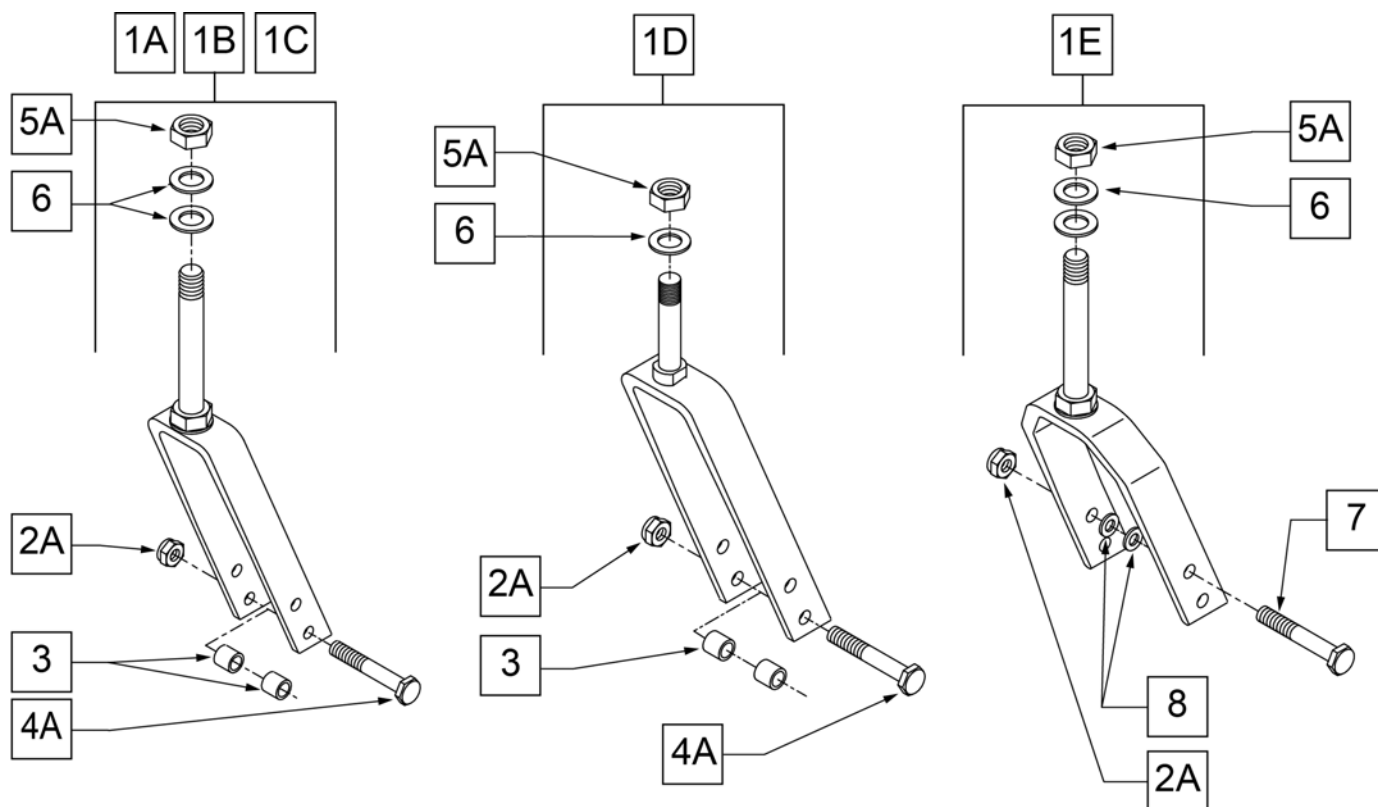

| Pos. | Item Number | Description                         | Remarks | Pos. | Item Number | Description                         | Remarks |
|------|-------------|-------------------------------------|---------|------|-------------|-------------------------------------|---------|
| 1A   | 780800      | 5 1/4" STD STEM W/ HDWR BLK         |         | 1C   | 780887      | 7 3/4" STD STEM PL L W/ HDWR BLK    | LEFT    |
| 1A   | 780802      | 5 1/4" STD STEM PL R W/ HDWR BLK    | RIGHT   | 1C   | 780892      | 7" QR W/ HDWR O/S                   |         |
| 1A   | 780803      | 5 1/4" STD STEM PL L W/ HDWR BLK    | LEFT    | 1D   | 780980      | 4 3/4" FORK STD STEM W/ HDWR BLK    |         |
| 1A   | 780804      | 5 1/4" + 3/4" STEM W/ HDWR BLK      |         | 1D   | 780981      | 4 3/4" STD STEM PL W/ HDWR R BLK    | RIGHT   |
| 1A   | 780806      | 5 1/4" + 3/4" STEM PL R W/ HDWR BLK | RIGHT   | 1D   | 780982      | 4 3/4" STEM PL W/ HDWR BLK          |         |
| 1A   | 780807      | 5 1/4" + 3/4" STEM PL L W/ HDWR BLK | LEFT    | 1D   | 780983      | 4 3/4"+3/4" STM PL W/HDWR R BLK     | RIGHT   |
| 1A   | 780812      | 5 1/4" QR W/ HDWR O/S               |         | 1D   | 780987      | 4 3/4" + 3/4" STEM PL L W/ HDWR BLK | LEFT    |
| 1B   | 780840      | 6" STD STEM W/ HDWR BLK             |         | 1E   | 787111      | 8" x 2" FORK STD STEM W/ HDWR       |         |
| 1B   | 780842      | 6" STD STEM PL R W/ HDWR BLK        | RIGHT   | 1E   | 787112      | 8" x 2 + 3/4" STEM W/ HDWR          |         |
| 1B   | 780843      | 6" STD STEM PL L W/ HDWR BLK        | LEFT    | 2A   | 106939      | 5/16-24 NYLOCK NUT THIN             |         |
| 1B   | 780844      | 6 + 3/4" STEM W/ HDWR BLK           |         | 2B   | 330005      | 5/16-24 NYLOCK NUT ALUMINUM         |         |
| 1B   | 780846      | 6 + 3/4" STEM PL R W/ HDWR BLK      | RIGHT   | 3    | 481113      | SPACER 5/16 x .380                  |         |
| 1B   | 780847      | 6 + 3/4" STEM PL L W/ HDWR BLK      | LEFT    | 4A   | 106987      | 5/16-24 x 2 1/2 HEX C/S             |         |
| 1B   | 780852      | 6" QR W/ HDWR O/S                   |         | 4B   | 330110      | 5/16-24 x 2 1/2 HEX C/S Ti          |         |
| 1C   | 780880      | 7" STD STEM W/ HDWR BLK             |         | 5A   | 106940      | 1/2-20 NYLOCK NUT                   |         |
| 1C   | 780882      | 7" STD STEM PL R W/ HDWR BLK        | RIGHT   | 5B   | 330010      | 1/2-20 NYLOCK NUT ALUMINUM          |         |
| 1C   | 780883      | 7" STD STEM PL L W/ HDWR BLK        | LEFT    | 6    | 106949      | 1/2 AN WASHER FLAT                  |         |
| 1C   | 780884      | 7 3/4" STEM W/ HDWR BLK             |         | 7    | 222253      | 5/16-24 x 3 1/4 HEX C/S             |         |
| 1C   | 780886      | 7 3/4" STD STEM PL R W/ HDWR BLK    | RIGHT   | 8    | 106947      | 5/16 AN WASHER FLAT                 |         |

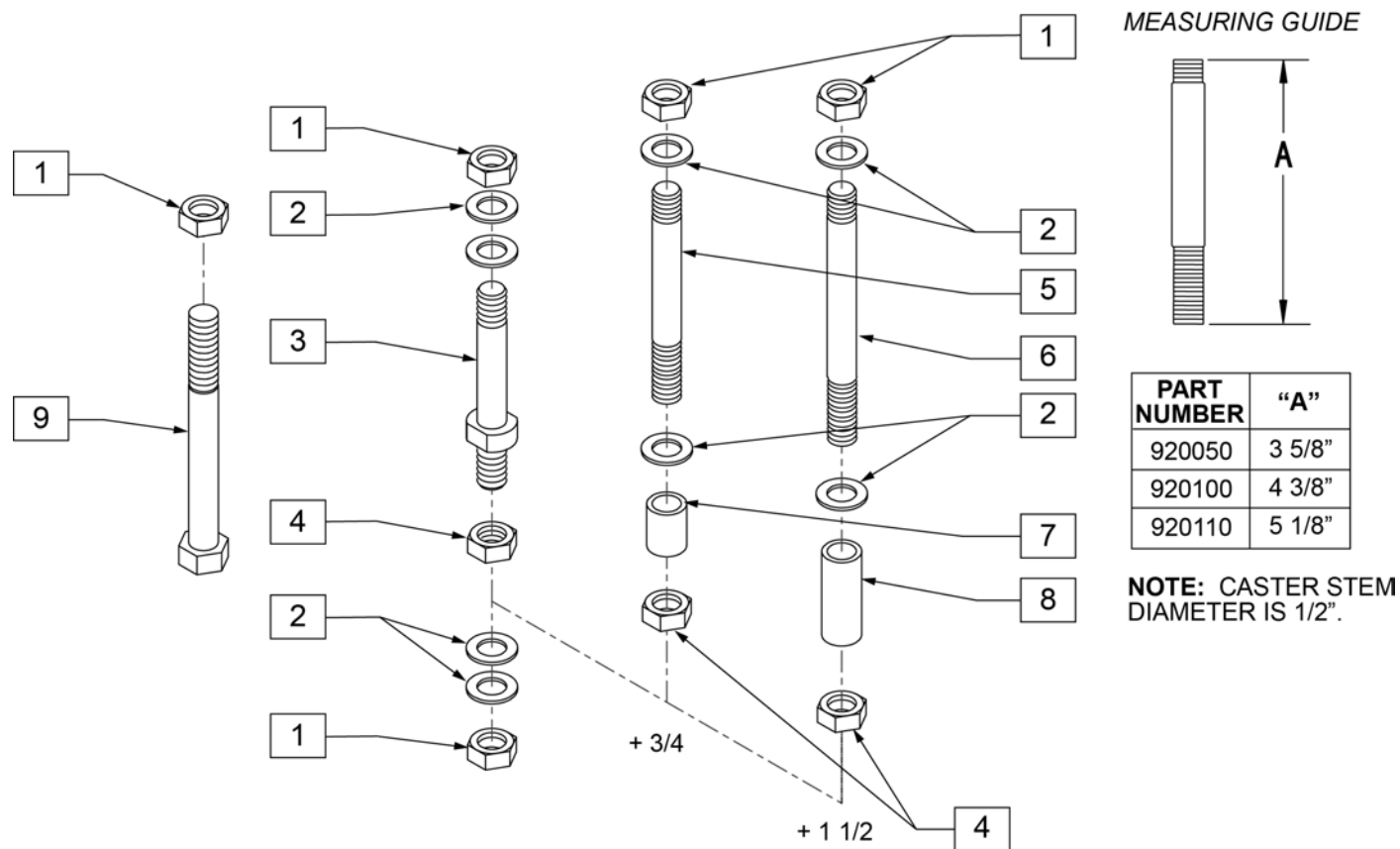


| Pos.          | Item Number | Description                          |
|---------------|-------------|--------------------------------------|
| 1             | 203100      | 1/2 - 20 NYLOCK NUT                  |
| 2             | 106949      | 1/2 AN WASHER FLAT                   |
| 4             | 203110      | 1/2 - 20 JAM NUT                     |
| 7             | 521560      | + 3/4" CASTER STEM SPACER BLK        |
| 8             | 521562      | + 1 1/2" CASTER STEM SPACER BLK      |
| 9             | 222292      | 1/2 - 20 x 4" GR 8 HEX C/S           |
| 1-3           | 920050      | REG CASTER STEM W/ HARDWARE          |
| 1, 2, 4, 5, 7 | 920100      | + 3/4" CASTER STEM W/ HARDWARE BLK   |
| 1, 2, 4, 6, 8 | 920110      | + 1 1/2" CASTER STEM W/ HARDWARE BLK |
| 1, 2, 9       | 920056      | BL CASTER STEM W/ HARDWARE           |

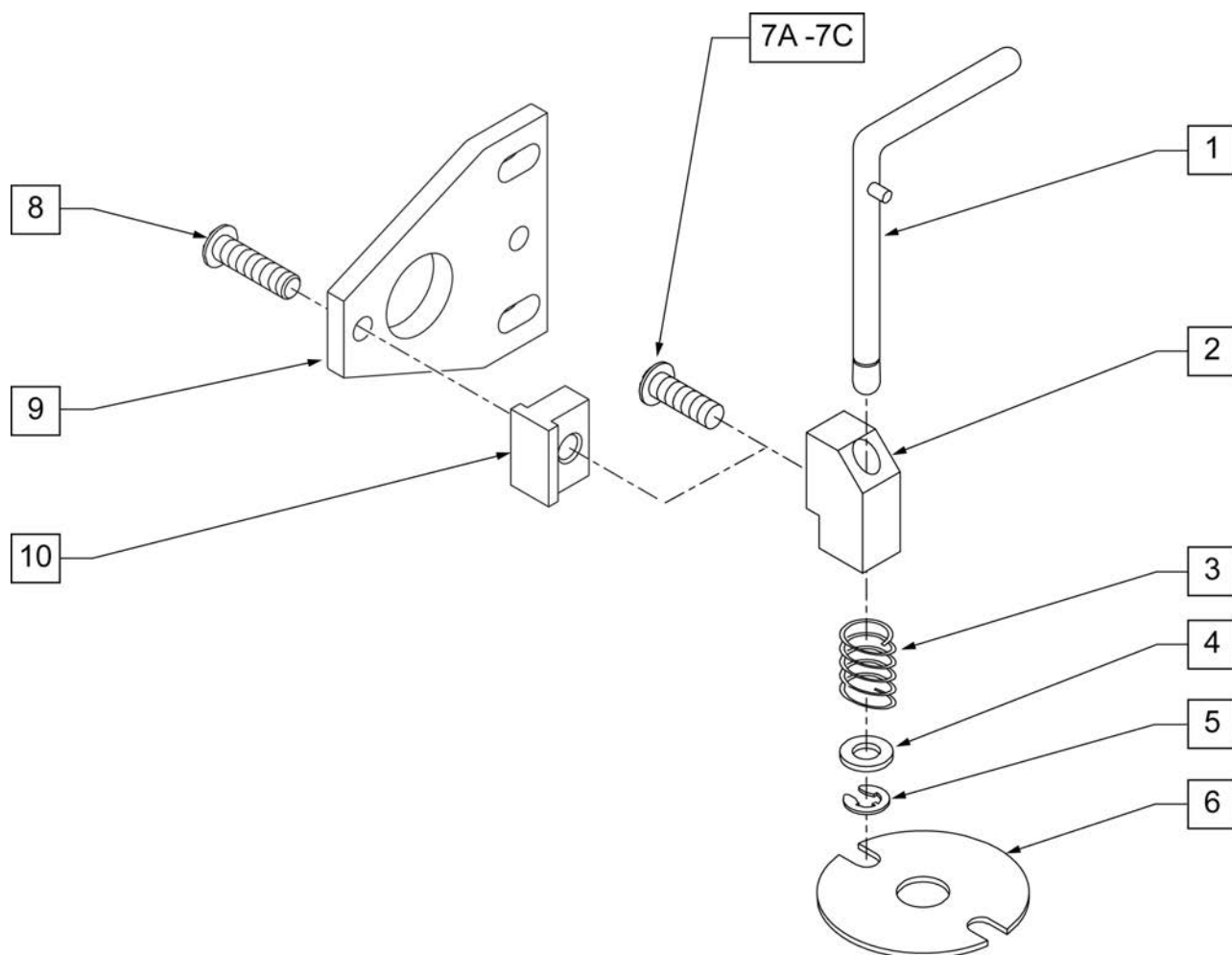


| Pos.   | Item Number | Description                                             | Remarks                           |
|--------|-------------|---------------------------------------------------------|-----------------------------------|
| 1      | 880761      | PIN LOCK ARM W/ PIN S/A                                 | RIGHT                             |
| 1      | 880762      | PIN LOCK ARM W/ PIN S/A                                 | LEFT                              |
| 2      | 500262      | CASTER PIN LOCK BLOCK S/A BLK                           |                                   |
| 3      | 321141      | .650L x 6 COIL SPRING                                   |                                   |
| 4      | 106943      | WASHER FLAT AN 1/4 ROHS                                 |                                   |
| 5      | 342000      | 1/4\" E-CLIP                                            |                                   |
| 6      | 505012      | CASTER PIN LOCK PLATE                                   |                                   |
| 7A     | 106959      | BHCS 1/4 -28 x 5/8 ROHS                                 | USED W/ S/A HANGER FRT FRAME ONLY |
| 7B     | 107015      | HHCS 1/4 -28 x 1 3/4 ROHS                               | USED W/ TRANSIT OPTION            |
| 7C     | 106973      | HHCS 1/4 -28 x 2 ROHS                                   | USED FOR GP FIXED FRT FRAME ONLY  |
| 8      | 106959      | 1/4 -28 X 5/8 BH C/S                                    |                                   |
| 9      | 505667      | PIN LOCK PLATE HI-ANGLE CASTER                          |                                   |
| 10     | 505666      | PIN LOCK BLOCK HI-ANGLE CASTER                          |                                   |
| 1-5    | 880750      | CASTER PIN LOCK S/A                                     | RIGHT                             |
| 1-5    | 880751      | CASTER PIN LOCK S/A                                     | LEFT                              |
| 1-6,7A | 900806      | CASTER PIN LOCK ASSM S/A OFF CHAIR                      | USED W/ S/A HANGER FRT FRAME ONLY |
| 1-6,7B | 900804      | CASTER PIN LOCK KIT TRIANGULAR CASTER NON-S/A OFF CHAIR | USED W/ TRANSIT OPTION            |
| 1-6,7C | 900807      | CASTER PIN LOCK ASSM GP/GPV                             | USED FOR GP FIXED FRT FRAME ONLY  |




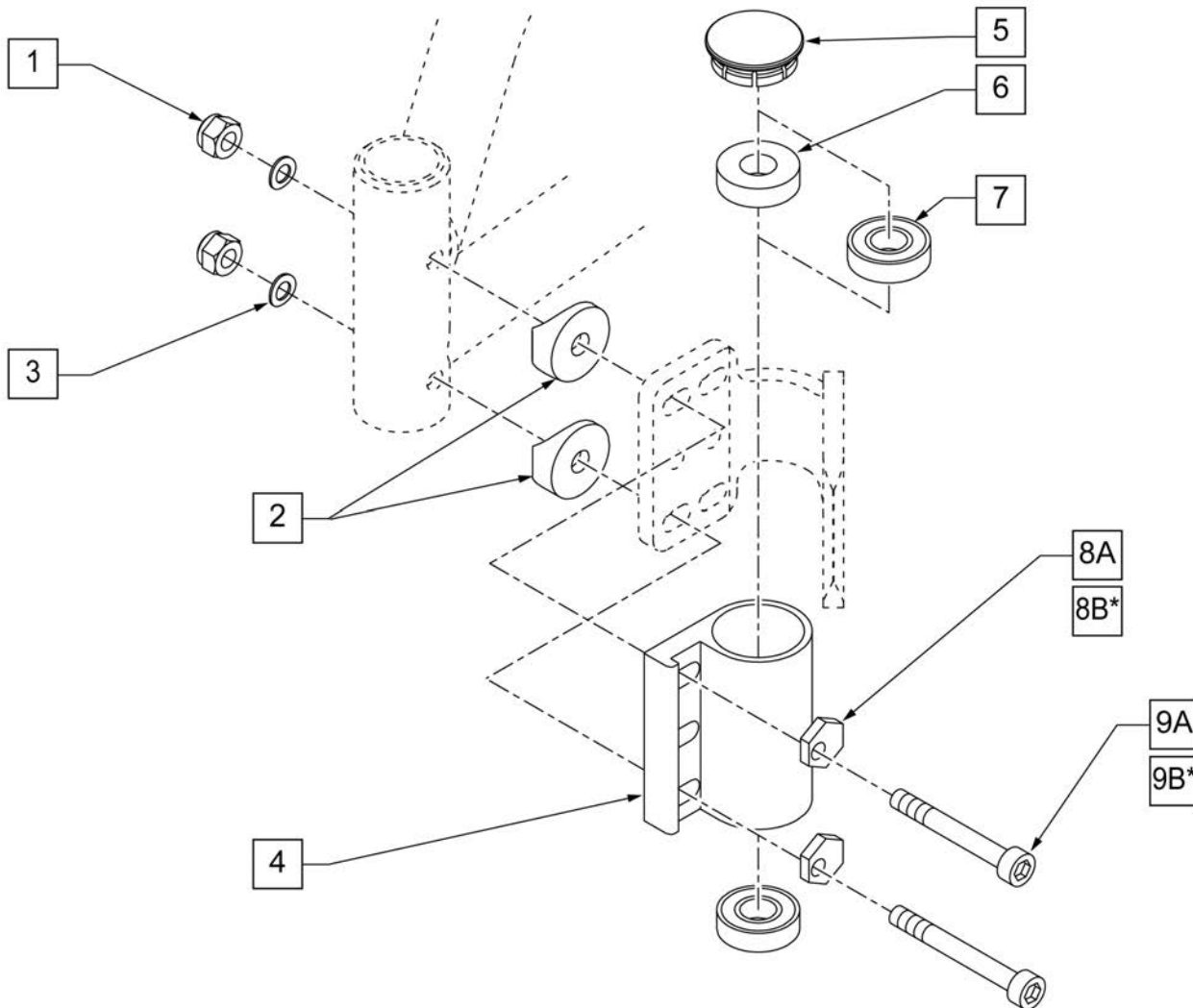

| Pos.       | Item Number | Description                     | Remarks |
|------------|-------------|---------------------------------|---------|
| 1          | 106941      | NUT AUTOMATION 1/4-28 ROHS      |         |
| 2          | 561086      | 1" HALF SADDLE 1/4" HOLE        |         |
| 3          | 104414      | WAVE WASHER 6 MM INORG          |         |
| 5          | 561070      | CASTER CAP PLASTIC              |         |
| 5          | 920028      | CASTER CAP PLASTIC              | PAIR    |
| 6          | 561050      | ANTI-FLUTTER BUSHING            |         |
| 7          | 349500      | 1 1/8 x 1/2 BEARING             |         |
| 8A         | 262128      | WASHER ECCENTRIC HD 1/4" HOLE   |         |
| *8B        | 481430      | ECCENTRIC WASHER TRANSPORT      |         |
| 9A         | 109809      | SHCS 1/4 -28 x 2 ROHS           |         |
| *9B        | 108791      | SHCS 1/4 -28 x 2 1/4 SHLDR ROHS |         |
| 4,6,7      | 780200      | CASTER PLATE SNGL BRNG BLK      | RIGHT   |
| 4,6,7      | 780201      | CASTER PLATE SNGL BRNG BLK      | LEFT    |
| 4,7        | 780210      | CASTER PLATE DBL BRNG BLK       | RIGHT   |
| 4,7        | 780212      | CASTER PLATE DBL BRNG BLK       | LEFT    |
| 1-3,5,8-10 | 920474      | CASTER PLATE HI ANGLE HDWR KIT  |         |

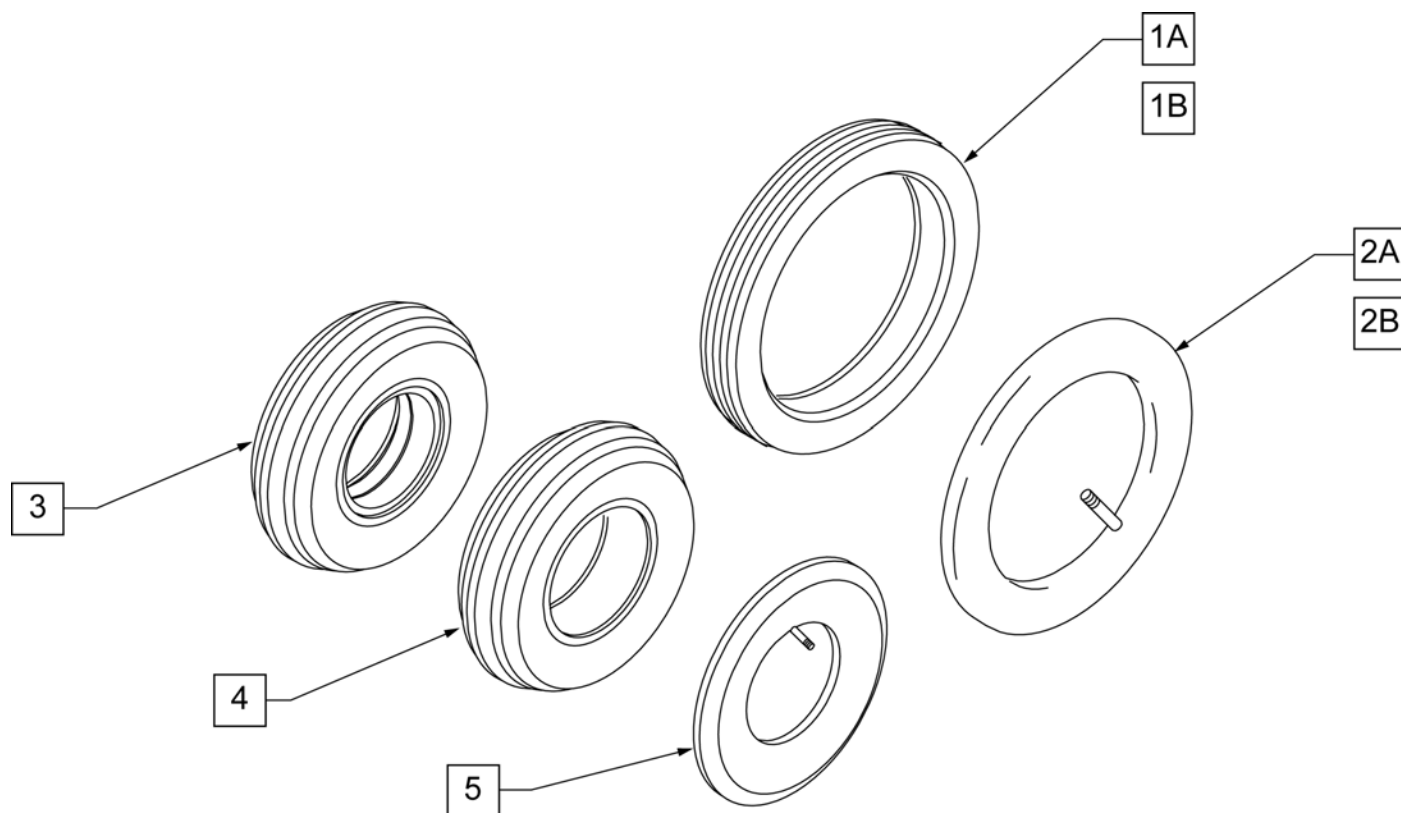


| Pos. | Item Number | Description                |
|------|-------------|----------------------------|
| 1A   | 383250      | 8" PNEUMATIC TIRE          |
| 1B   | 383240      | 6" PNEUMATIC TIRE          |
| 2A   | 382250      | 8" TUBE                    |
| 2B   | 382245      | 6" TUBE                    |
| 3    | 384701      | TIRE FOAMED INSERT 8" x 2" |
| 4    | 384700      | TIRE PNEUMATIC 8" x 2"     |
| 5    | 384730      | INNER TUBE CASTER 8" x 2"  |
| 1A,3 | 383255      | 8" PNEUMATIC TIRE AND TUBE |
| 1B   |             | Part 1B                    |
| 1A   |             | Part 1A                    |
| 2B   |             | Part 2B                    |
| 2A   |             | Part 2A                    |
| 3    |             | Part 3                     |
| 4    |             | Part 4                     |
| 5    |             | Part 5                     |

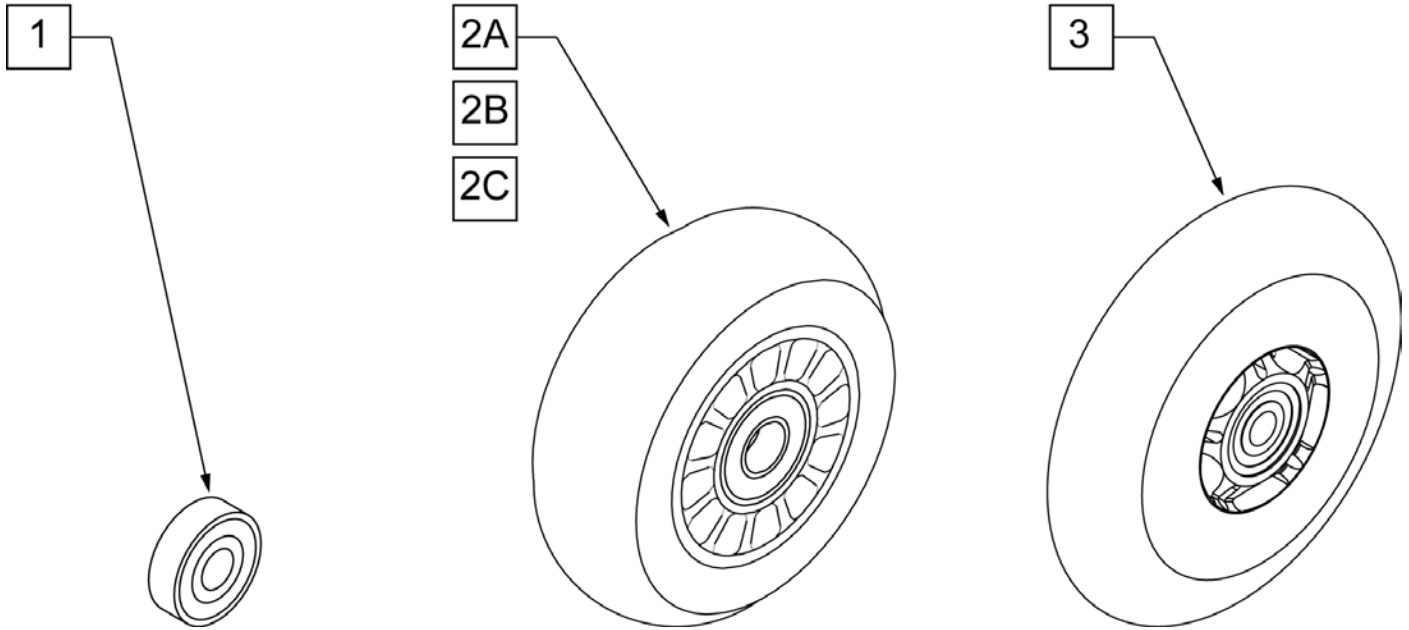


| Pos. | Item Number | Description                      | HCPC  | Remarks          |
|------|-------------|----------------------------------|-------|------------------|
| 1    | 349502      | 7/8"x 5/16" BEARING              |       |                  |
| 2A   | 400108      | 4" MICRO CASTER LIGHTED          | K0077 | 4" MICRO LIGHTED |
| 2B   | 920025      | 3" MICRO CASTER W/ BEARING -PAIR | K0077 | PAIR 3" MICRO    |
| 2C   | 400107      | 3" MICRO CASTER LIGHTED          | K0077 | 3" MICRO LIGHTED |
| 3    | 052129      | 5" MICRO CASTER LIGHTED          | K0077 | 5" MICRO LIGHTED |


NOTE: 7" AND 8" CASTERS ARE NOT AVAILABLE ON THE Q2 LITE.

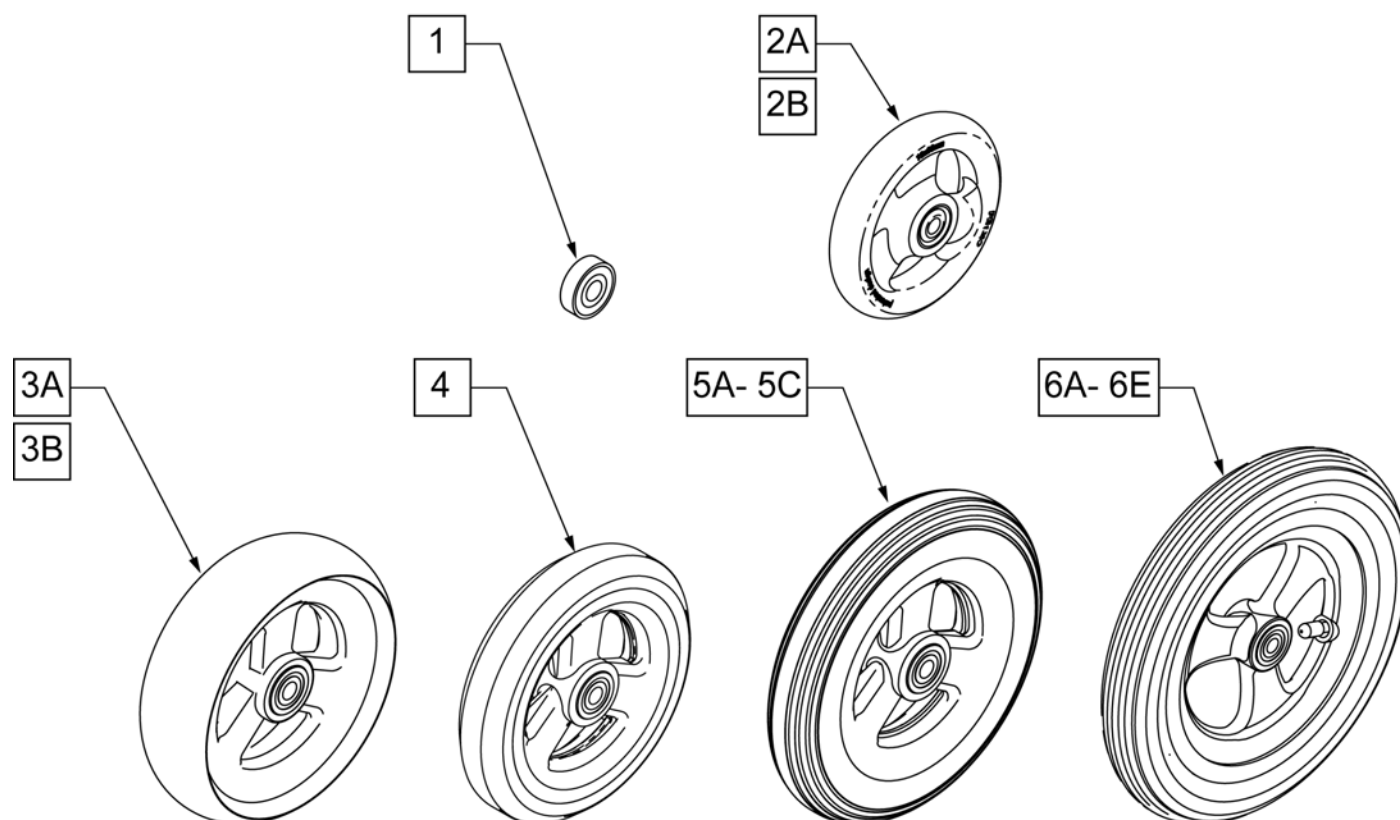


| Pos. | Item Number | Description                    | Remarks                  |
|------|-------------|--------------------------------|--------------------------|
| 1    | 349502      | 7/8"x 5/16" BEARING            |                          |
| 2A   | 101957      | 4"x1-1/4" POLY 3 SPOKE CSTR    | 4" X 1.25 SEMI-PNEUMATIC |
| 2B   | 101098      | CASTER 4" PLASTIC/POLY W/BRG   | 4" LOW PROFILE POLY      |
| 3A   | 101259      | 5"x1-1/2" POLY 3 SPOKE CSTR    | 5" x 1.5" SEMI-PNEUMATIC |
| 3B   | 101255      | 6"x1-1/2" POLY 3 SPOKE CSTR    | 6" x 1.5" SEMI-PNEUMATIC |
| 4    | 101254      | 5" LOW POLY 3 SPOKE CASTER     | 5" LOW PROFILE POLY      |
| 5A   | 101256      | 6" SEMI PNEU 3 SPOKE CASTER    | 6" POLY                  |
| 5B   | 101257      | 6" PNEUMATIC 3 SPOKE CASTER    | 6" PNEUMATIC             |
| 5C   | 383245      | 6" PNEUMATIC TIRE & TUBE       |                          |
| 6A   | 400212      | CASTER 8" PNEU 3 SPK ASSM      | 8" PNEUMATIC             |
| 6B   | 400213      | CASTER 8" POLY 3 SPK ASSM      | 8" POLY                  |
| 6C   | 400214      | CASTER 8" LOW POLY 3 SPK ASSM  | 7" x LOW PROFILE POLY    |
| 6D   | 383255      | 8" PNEUMATIC TIRE & TUBE       |                          |
| 6E   | 400215      | CASTER 8" WIDE POLY 3 SPK ASSM | 8" x 1.5" LOW POLY       |

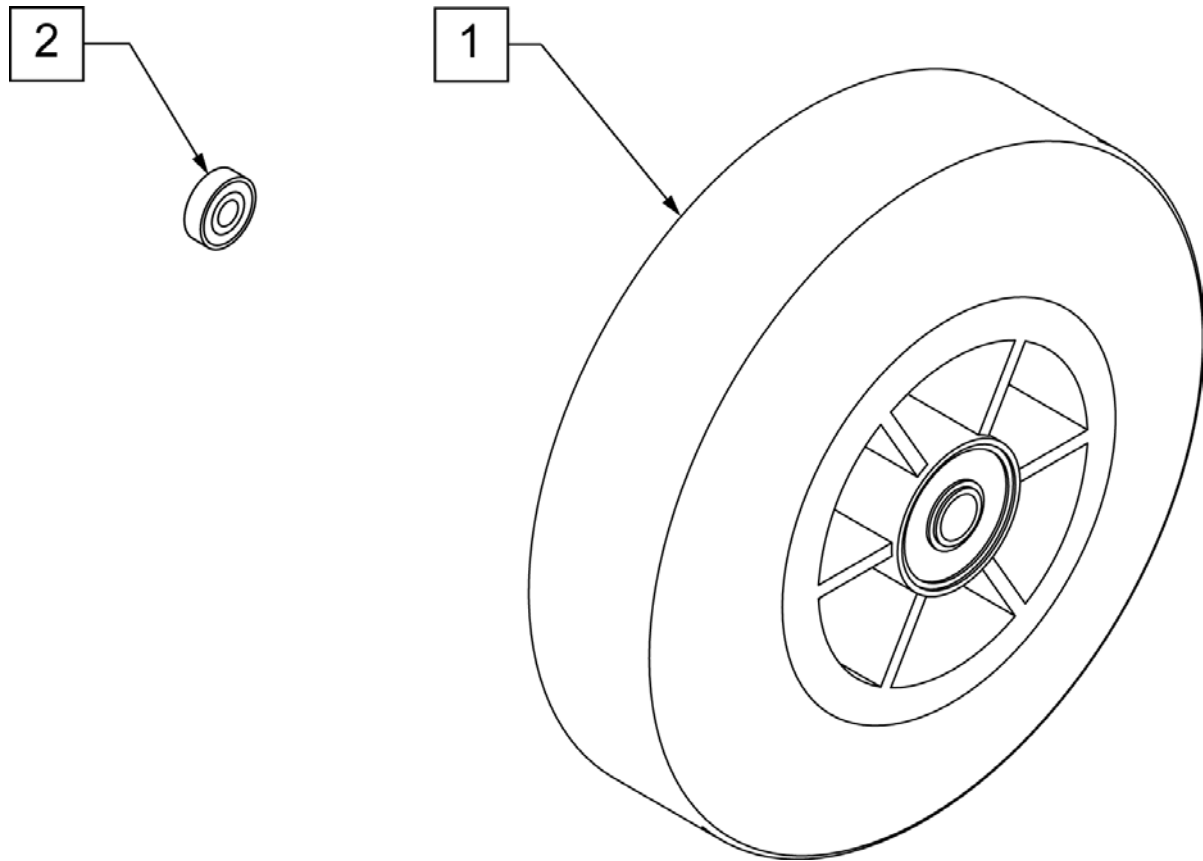


| Pos. | Item Number | Description                 | HCPC  | Remarks                       |
|------|-------------|-----------------------------|-------|-------------------------------|
| 1    | 400103      | CASTER 5" SOFT ROLL W/BRNGS | K0077 | 5" SOFT ROLL SEMI PNEU (GREY) |
| 2    | 349502      | 7/8"x 5/16" BEARING         |       |                               |

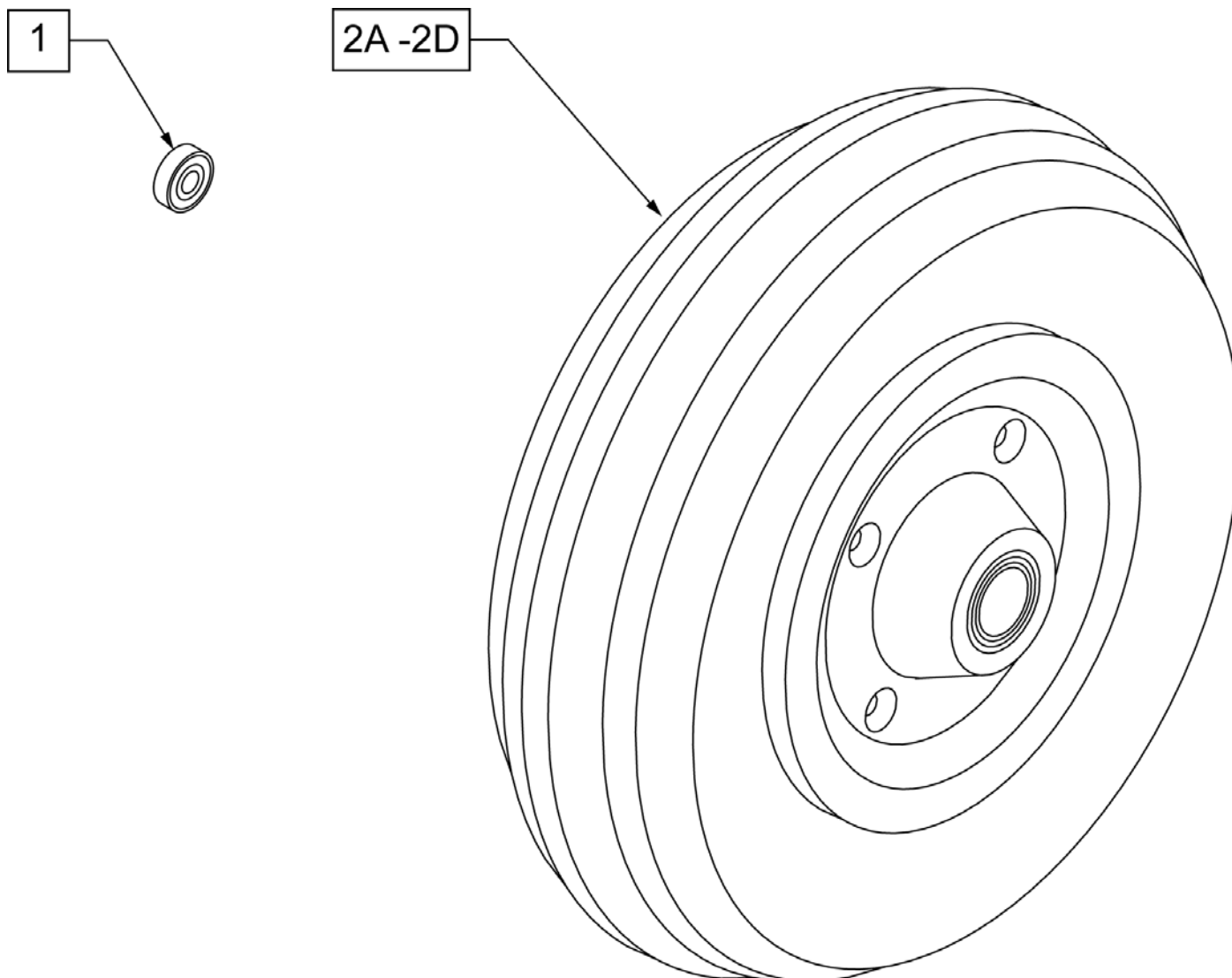

| Pos. | Item Number | Description                    | Remarks                     |
|------|-------------|--------------------------------|-----------------------------|
| 1    | 349502      | 7/8"x 5/16" BEARING            |                             |
| 2A   | 400700      | 8" X 2" CMPST W/PNEU & BEARING | 8" x 2" PNEUMATIC           |
| 2B   | 400702      | 8" X 2" CMPST W/PNEU INSRT&BRG | 8" x 2" PNEUMATIC W/ INSERT |
| 2C   | 384700      | TIRE PNEUMATIC 8" x 2"         |                             |
| 2D   | 384730      | INNER TUBE CASTER 8" x 2"      |                             |

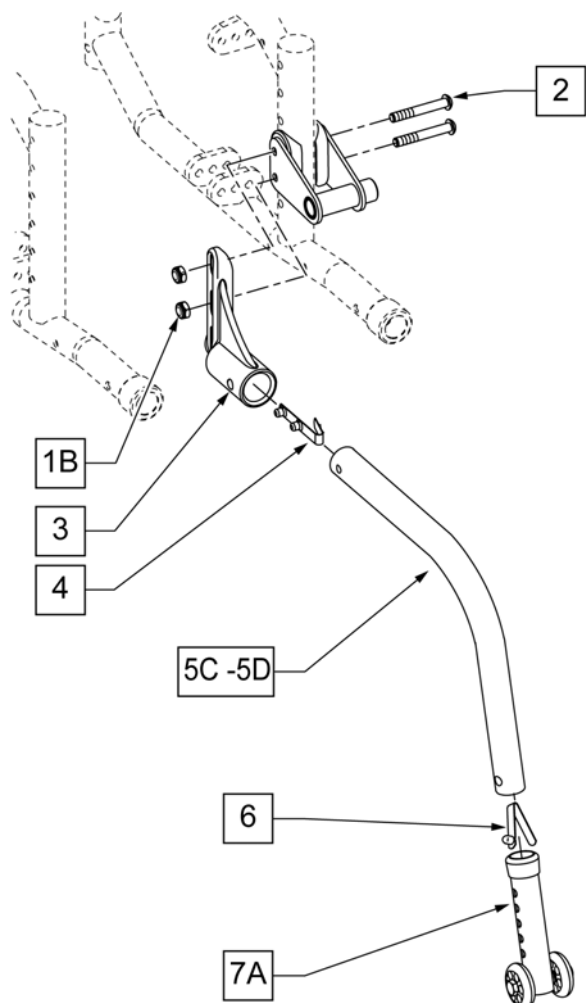

For use w/ 18-24" Wheels

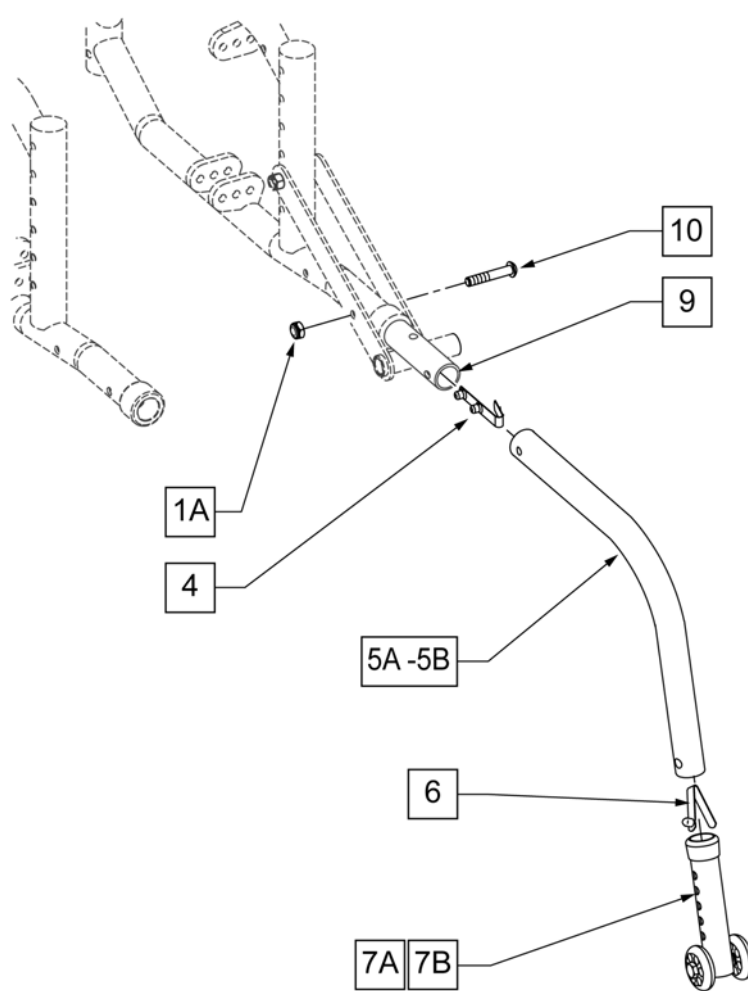


For use w/ 12" &amp; 16" Wheels


| Pos.      | Item Number | Description                           | Remarks                                       |
|-----------|-------------|---------------------------------------|-----------------------------------------------|
| 1A        | 106935      | NUT NYLOCK THIN 1/4 -28 ROHS          |                                               |
| 1B        | 106934      | NUT NYLOCK 1/4 -28 ROHS               |                                               |
| 2         | 106966      | BHCS 1/4 -28 x 2 1/4 ROHS             |                                               |
| 3         | 920407      | ANTI-TIP RECEIVERS                    | PAIR                                          |
| 4         | 301109      | SNAP BUTTON DUAL ONE SIDE             |                                               |
| 4,5A,6    | 880985      | ANTI TIP TUBE TS W/ SNAP BUTTON       | FOR USE W/ 12" REAR WHEELS                    |
| 4,5B,6    | 882898      | ANTI TIP TUBE KDZ R BL W/ SNAP BUTTON | RIGHT -FOR USE W/ 16" REAR WHEELS             |
| 4,5B,6    | 882899      | ANTI TIP TUBE KDZ L BL W/ SNAP BUTTON | LEFT -FOR USE W/ 16" REAR WHEELS              |
| 4,5C,6    | 882966      | ANTI TIP TUBE X/S R W/ SNAP BUTTON    | RIGHT -FOR USE W/ 18", 20", 22" REAR WHEELS   |
| 4,5C,6    | 882967      | ANTI TIP TUBE X/S L W/ SNAP BUTTON    | LEFT -FOR USE W/ 18", 20", 22" REAR WHEELS    |
| 4,5D,6    | 882974      | ANTI TIP TUBE EXTND REG R W/ SNAP     | RIGHT -FOR USE W/ 24" REAR WHEELS             |
| 4,5D,6    | 882975      | ANTI TIP TUBE EXTND REG L W/ SNAP     | LEFT -FOR USE W/ 24" REAR WHEELS              |
| 6         | 301100      | SNAP BUTTON 1/4" SINGLE               |                                               |
| 7A        | 881512      | ANTI-TIP SLEEVE W/WHEELS BLK (4H)     | USED FOR 16", 18", 20", 22" & 24" REAR WHEELS |
| 7B        | 880486      | ANTI TIP SLEEVE W/ WHLS BZ/ZP         | USED FOR 12" WHEELS ONLY                      |
| 8         | 561004      | 1" SADDLE 1/4" HOLE                   |                                               |
| 9         | 109734      | TUBE EXTENSION FRAME                  |                                               |
| 10        | 112390      | BHCS 1/4 -28 x 1 3/4 FULL ROHS        |                                               |
| 4,5A,6,7B | 880510      | ANTI TIP TUBE TS 12" WHEEL            | For 12" WHEELS                                |

| Pos.      | Item Number | Description                    | Remarks                         |
|-----------|-------------|--------------------------------|---------------------------------|
| 4,5A,6,7B | 900705      | ANTI TIP ASSM TS 12" WHEEL     | PAIR                            |
| 4,5B,6,7A | 882904      | ANTI TIP ASSM KIDZ R BLACK     | For 16" WHEELS -RIGHT           |
| 4,5B,6,7A | 882905      | ANTI TIP ASSM KIDZ L BLACK     | For 16' WHEELS -LEFT            |
| 4,5B,6,7A | 900733      | ANTI TIP ASSM KIDZ BLK         | PAIR                            |
| 4,5C,6,7A | 882908      | ANTI TIP ASSM SMALL EXT R BLK  | For 18", 20",22" WHEELS - RIGHT |
| 4,5C,6,7A | 882909      | ANTI TIP ASSM SMALL EXT L BLK  | For 18", 20",22" WHEELS - LEFT  |
| 4,5C,6,7A | 900696      | ANTI TIP ASSM SMALL EXTEND     | PAIR                            |
| 4,5D,6,7A | 882910      | ANTI TIP ASSM REG EXTEND R BLK | For 24" WHEELS -RIGHT           |
| 4,5D,6,7A | 882911      | ANTI TIP ASSM REG EXTEND L BLK | For 24" WHEELS -LEFT            |
| 4,5D,6,7A | 900697      | ANTI-TIP ASSM REGULAR EXTEND   | PAIR                            |
| 1B,2,3    | 115730      | A/T REC W/ HDW KIT R           | RIGHT                           |
| 1B,2,3    | 115731      | A/T REC W/ HDW KIT L           | LEFT                            |




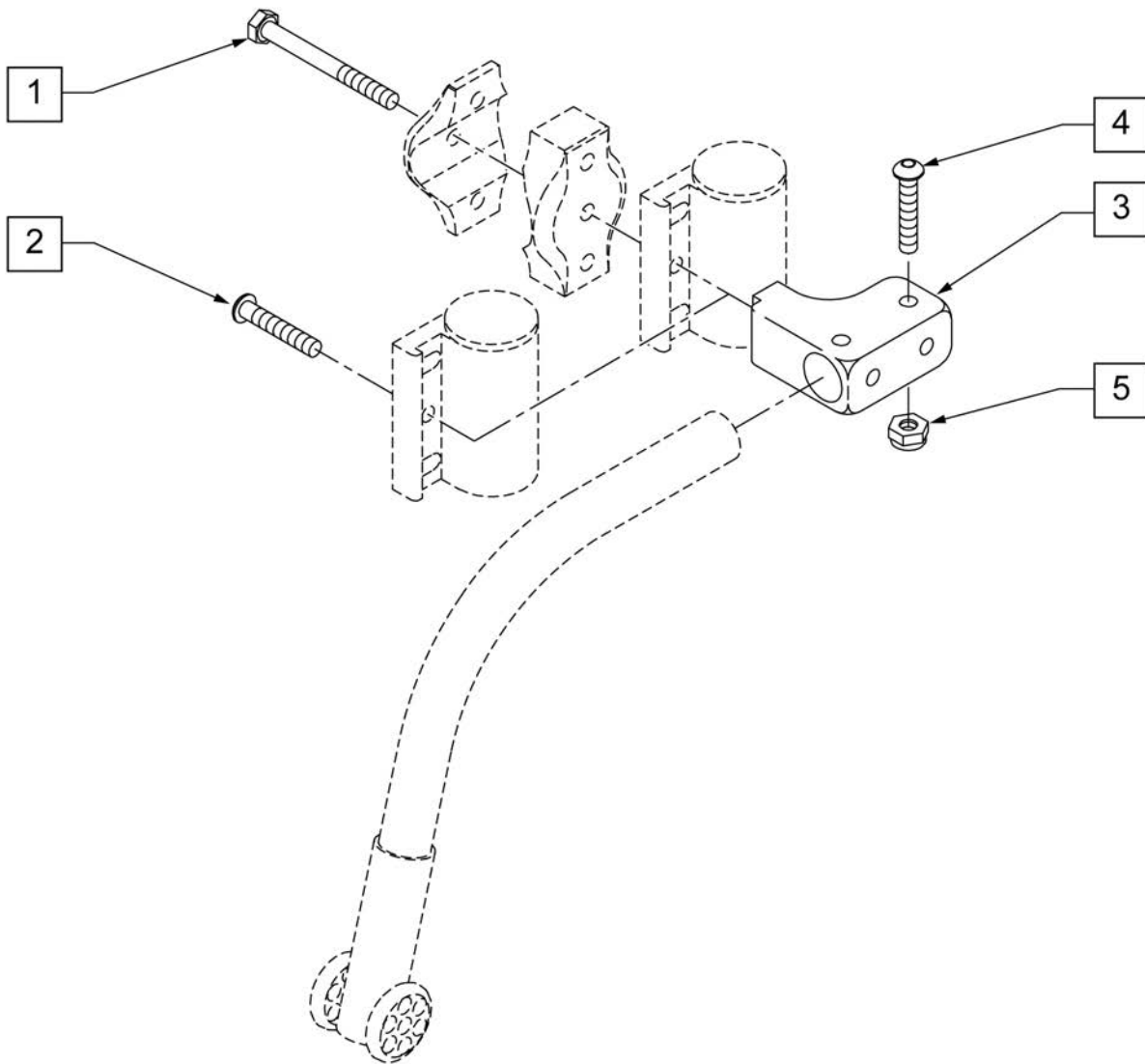

| Pos. | Item Number | Description                     | Remarks |
|------|-------------|---------------------------------|---------|
| 1    | 106973      | HHCS 1/4 -28 x 2 ROHS           |         |
| 2    | 106961      | BHCS 1/4 -28 x 1 1/4 ROHS       |         |
| 3    | 505340      | ANTI-TIP RECEIVER BLOCK FRONT   |         |
| 4    | 113283      | BHCS 10 -32 x 1 1/8 ROHS        |         |
| 5    | 106930      | NUT NYLOCK 10 -32 ROHS          |         |
| 1-5  | 920408      | FRONT ANTI-TIP REC CASTER MOUNT | PAIR    |

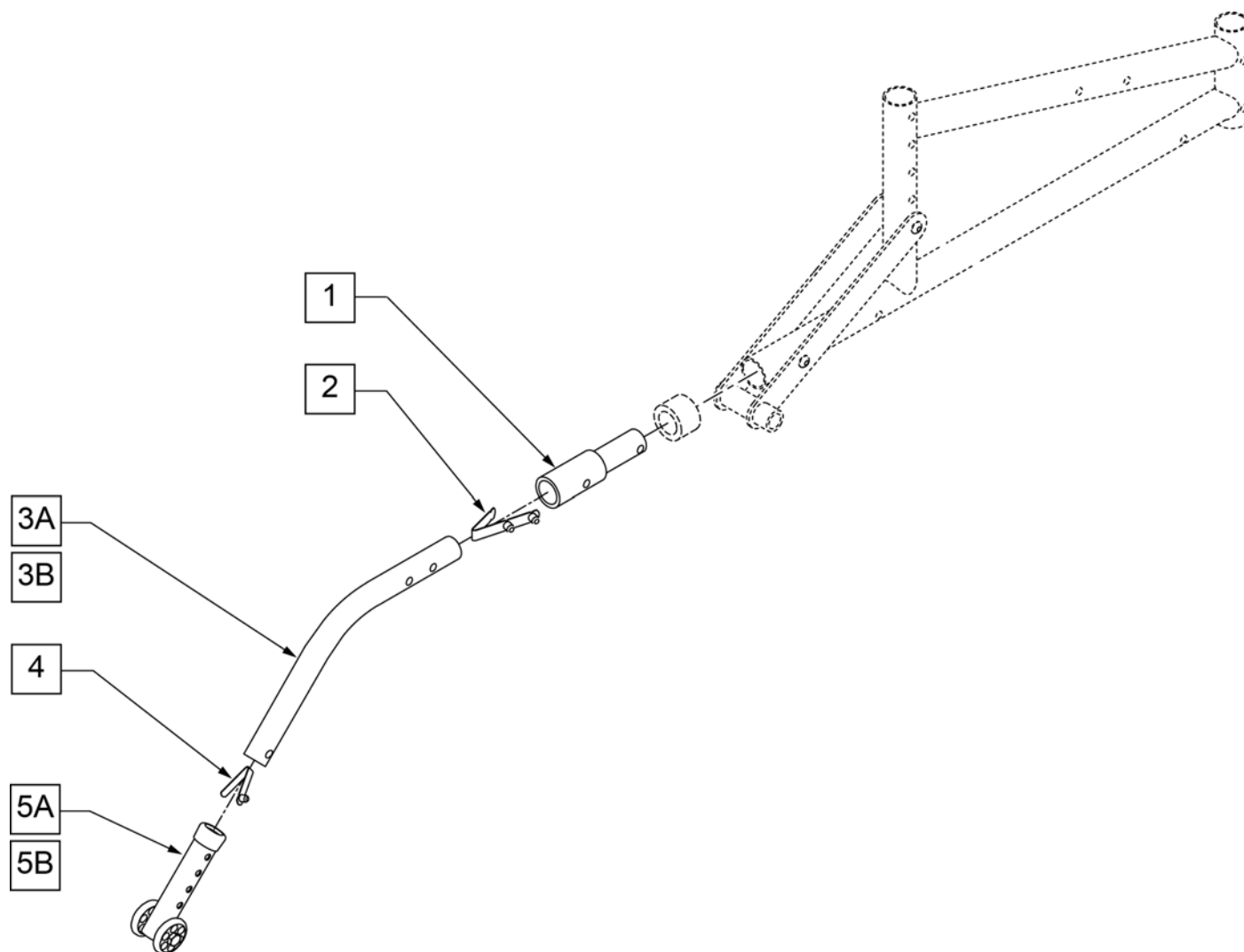


| Pos.               | Item Number | Description                         | Remarks |
|--------------------|-------------|-------------------------------------|---------|
| 1                  | 483097      | ANTI-TIP INSERT 12" WHEEL           |         |
| 2                  | 301109      | SNAP BUTTON DUAL ONE SIDE           |         |
| 2,3A,4             | 880985      | ANTI-TIP TUBE TS W/ SNAP BUTTON     |         |
| 2,3B,4             | 882978      | ANTI-TIP TB KIDZ R W/ SNAP BUTTON   | RIGHT   |
| 2,3B,4             | 882979      | ANTI-TIP TB KIDZ L W/ SNAP BUTTON   | LEFT    |
| 4                  | 301100      | 1/4" SNAP BUTTON SINGLE             |         |
| 5A                 | 880486      | ANTI-TIP SLEEVE W/ WHEELS BZ/ZP     |         |
| 5B                 | 881512      | ANTI-TIP SLEEVE W/ WHEELS BLK 4H    |         |
| 2,4A,4,5A          | 880510      | ANTI-TIP TUBE TS 12" WHEEL          |         |
| 2,3B,4,5B          | 900705      | ANTI-TIP ASSEMBLY TS 12" WHEEL (PR) | PAIR    |
| 5,7,8A,9,10A,11-15 | 882904      | ANTI-TIP ASSEMBLY BLK KIDZ          | RIGHT   |
| 5,7,8A,9,10A,11-15 | 882905      | ANTI-TIP ASSEMBLY BLK KIDZ          | LEFT    |

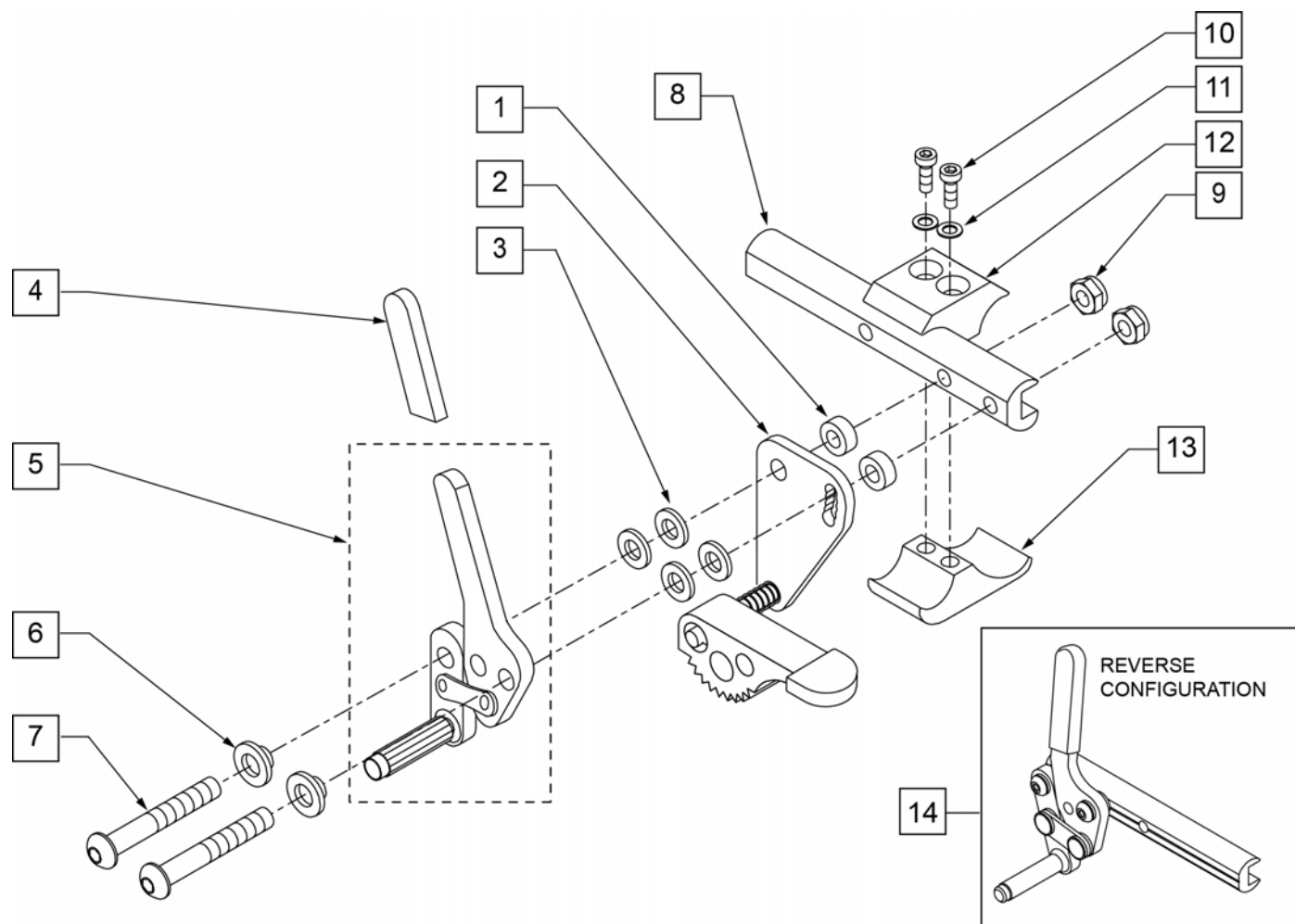


| Pos.  | Item Number | Description                         | Remarks |
|-------|-------------|-------------------------------------|---------|
| 1     | 483027      | 1/4 x 1/4 ALUM SPACER BLK           |         |
| 2     | 880904      | GRADE AID PLATE W/WHEEL BLK         | RIGHT   |
| 2     | 880905      | GRADE AID PLATE W/WHEEL BLK         | LEFT    |
| 3     | 561028      | 1/4" NYLON WASHER                   |         |
| 4     | 560500-QU   | HB PLASTIC COVER                    |         |
| 4,5   | 881748      | WHEEL LOCK ASSM PUSH                | RIGHT   |
| 4,5   | 881749      | WHEEL LOCK ASSM PUSH                | LEFT    |
| 6     | 560126      | .250 ID .312 OD BUSHING PLST        |         |
| 7     | 226125      | 1/4-28 X 1 1/8 BH C/S               |         |
| 7     | 226130      | 1/4-28 x 1 1/2 BH C/S (W/GRADE AID) |         |
| 8     | 550447      | WHL LOCK SLIDE BLOCK LONG           |         |
| 9     | 106941      | 1/4-28 AUTOMATION NUT               |         |
| 10    | 107032      | 1/4-28 x 1 SOC C/S BLK              |         |
| 11    | 262114      | 1/4 WASHER HI COLLAR LOCK ZN        |         |
| 12    | 550659      | CLAMP 1 1/4" RECESS HALF BLK        |         |
| 13    | 550658      | CLAMP 1 1/4" THREADED BLK           |         |
| 10-13 | 107639      | WHL LCK CLMP ASM                    |         |
| 3-9   | 881946      | WHEEL LOCK PUSH LARGE W/O CLAMP     | RIGHT   |
| 3-9   | 881947      | WHEEL LOCK PUSH LARGE W/O CLAMP     | LEFT    |
| 1-9   | 882023      | WHL LOCK PUSH LARGE W/GRADE AID     | RIGHT   |
| 1-9   | 882024      | WHL LOCK PUSH LARGE W/GRADE AID     | LEFT    |

| Pos. | Item Number | Description           | Remarks |
|------|-------------|-----------------------|---------|
| 14   | 120041      | WHL LCK PUSH LG R REV | RIGHT   |
| 14   | 120042      | WHL LCK PUSH LG L REV | LEFT    |

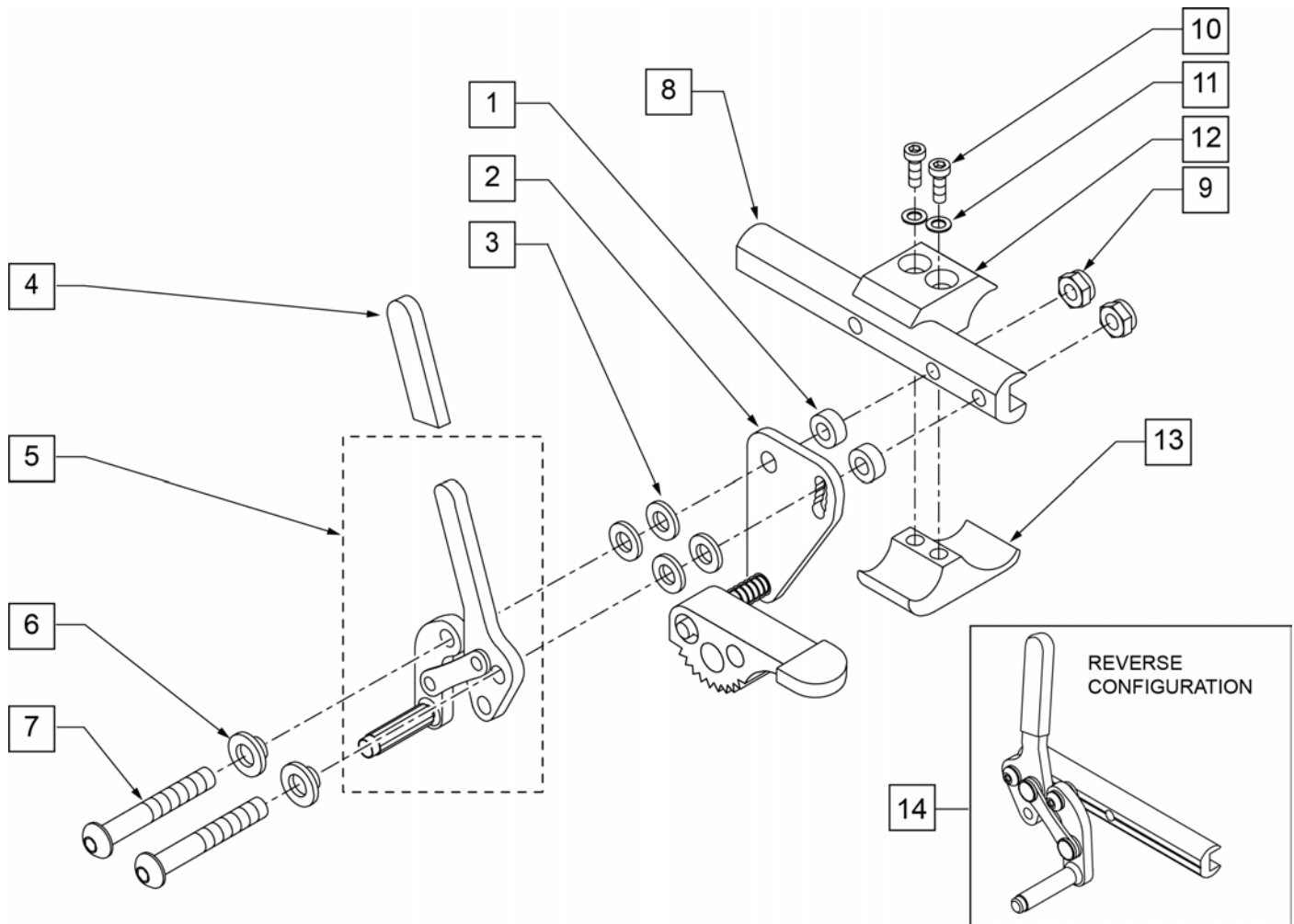

| Pos.  | Item Number | Description                         | Remarks |
|-------|-------------|-------------------------------------|---------|
| 1     | 483027      | 1/4 x 1/4 ALUM SPACER BLK           |         |
| 2     | 880904      | GRADE AID PLATE W/ WHEEL BLK        | RIGHT   |
| 2     | 880905      | GRADE AID PLATE W/ WHEEL BLK        | LEFT    |
| 3     | 561028      | 1/4" NYLON WASHER                   |         |
| 4     | 560500-QU   | HB PLASTIC COVER                    |         |
| 4,5   | 881798      | WHEEL LOCK ASSM PULL                | RIGHT   |
| 4,5   | 881799      | WHEEL LOCK ASSM PULL                | LEFT    |
| 6     | 560126      | .250 ID .312 OD BUSHING PLST        |         |
| 7     | 226110      | 1/4-28 x 1 BH C/S                   |         |
| 7     | 226130      | 1/4-28 x 1 1/2 BH C/S (W/GRADE AID) |         |
| 8     | 550447      | WHL LCK SLIDE BLOCK LNG             |         |
| 9     | 106935      | NUT NYLOCK THIN 1/4-28 ROHS         |         |
| 10    | 226112      | 1/4-28 x 1 SOC C/S BLK              |         |
| 11    | 262114      | 1/4 WASHER HI COLLAR LOCK ZN        |         |
| 12    | 550659      | CLAMP 1 1/4" RECESS HALF BLK        |         |
| 13    | 550658      | CCLAMP 1 1/4" THREADED BLK          |         |
| 10-13 | 107639      | WHL LCK CLMP ASM                    |         |
| 3-9   | 881948      | WHL LOCK PULL LARGE W/O CLMP        | RIGHT   |
| 3-9   | 881949      | WHL LOCK PULL LARGE W/O CLMP        | LEFT    |
| 1-9   | 882025      | WHL LOCK PULL LARGE W/GRADE AID     | RIGHT   |


| Pos. | Item Number | Description                     | Remarks |
|------|-------------|---------------------------------|---------|
| 1-9  | 882026      | WHL LOCK PULL LARGE W/GRADE AID | LEFT    |
| 14   | 120043      | WHL LCK PULL LG R REV           | RIGHT   |
| 14   | 120044      | WHL LCK PULL LG L REV           | LEFT    |

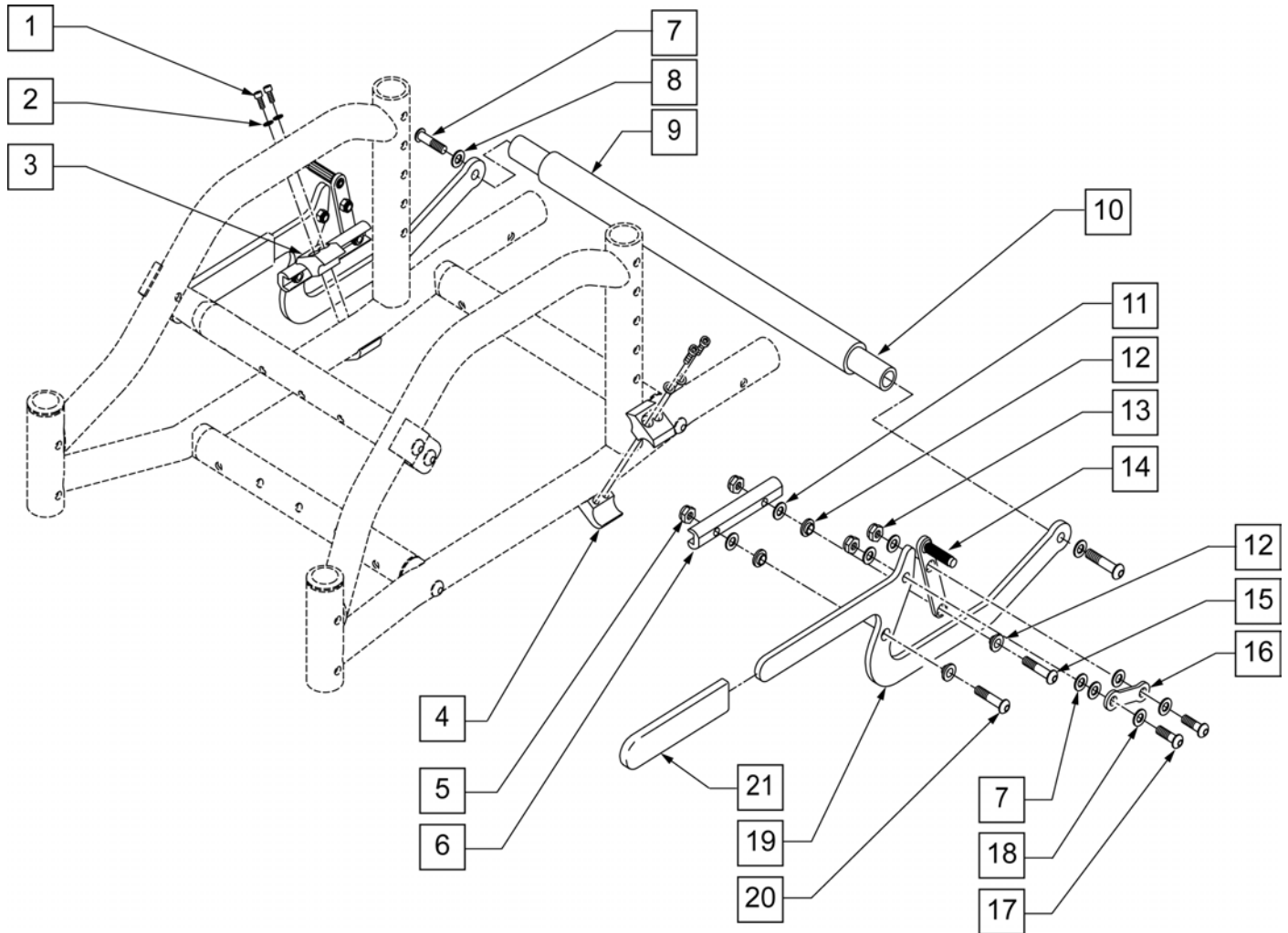


| Pos. | Item Number | Description                  | Pos. | Item Number | Description                  |
|------|-------------|------------------------------|------|-------------|------------------------------|
| 1    | 226112      | 1/4-28 x 1 SOC C/S BLK       | 11   | 561028      | 1/4" NYLON WASHER            |
| 2    | 262114      | 1/4 WASHER HI COLLAR LOCK ZN | 12   | 262115      | 1/4" NYLON FLANGE BUSHING    |
| 3    | 550659      | CLAMP 1 1/4" RECESS HALF BLK | 13   | 106935      | NUT NYLOCK THIN 1/4-28 ROHS  |
| 4    | 550658      | CLAMP 1 1/4" THREADED BLK    | 14   | 880531      | LOCK TOGGLE AND GRIP         |
| 5    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS  | 15   | 226122      | 1/4-28 x 1.10 BH C/S         |
| 6    | 550268      | WHEEL LOCK FOOT SLIDE BLOCK  | 16   | 504995      | TOGGLE LINK 1.125            |
| 7    | 226061      | 1/4-20 x 1 BH C/S W/PATCH    | 17   | 226100      | 1/4-28 x 3/4 BH C/S          |
| 8    | 106944      | WASHER FLAT THIN AN 1/4 ROHS | 18   | 262130      | 6mm WASHER WAVE              |
| 9    | 560612      | 8 x .705 x .188 FOAM TUBE    | 19   | 505153      | FOOT WHEEL LOCK LEVER        |
| 9,10 | 880710      | FOOTLOCK TUBE W/FOAM 10"     | 20   | 226110      | 1/4-28 x 1 BH C/S            |
| 9,10 | 880711      | FOOTLOCK TUBE W/FOAM 11"     | 21   | 560305      | 4 x .375 ID x .125 FOAM GRIP |
| 9,10 | 880712      | FOOTLOCK TUBE W/FOAM 12"     | 1-4  | 107639      | WHL LCK CLMP ASM             |
| 9,10 | 880713      | FOOTLOCK TUBE W/FOAM 13"     | 1-21 | 900740      | FOOT WHEEL LOCK ASSM 10"     |
| 9,10 | 880714      | FOOTLOCK TUBE W/FOAM 14"     | 1-21 | 900741      | FOOT WHEEL LOCK ASSM 11"     |
| 9,10 | 880715      | FOOTLOCK TUBE W/FOAM 15"     | 1-21 | 900742      | FOOT WHEEL LOCK ASSM 12"     |
| 9,10 | 880716      | FOOTLOCK TUBE W/FOAM 16"     | 1-21 | 900743      | FOOT WHEEL LOCK ASSM 13"     |
| 9,10 | 880717      | FOOTLOCK TUBE W/FOAM 17"     | 1-21 | 900744      | FOOT WHEEL LOCK ASSM 14"     |
| 9,10 | 880718      | FOOTLOCK TUBE W/FOAM 18"     | 1-21 | 900745      | FOOT WHEEL LOCK ASSM 15"     |
| 9,10 | 880719      | FOOTLOCK TUBE W/FOAM 19"     | 1-21 | 900746      | FOOT WHEEL LOCK ASSM 16"     |
| 9,10 | 880725      | FOOTLOCK TUBE W/FOAM 20"     | 1-21 | 900747      | FOOT WHEEL LOCK ASSM 17"     |

| Pos. | Item Number | Description              | Pos. | Item Number | Description              |
|------|-------------|--------------------------|------|-------------|--------------------------|
| 1-21 | 900748      | FOOT WHEEL LOCK ASSM 18" | 1-21 | 900750      | FOOT WHEEL LOCK ASSM 20" |
| 1-21 | 900749      | FOOT WHEEL LOCK ASSM 19" |      |             |                          |



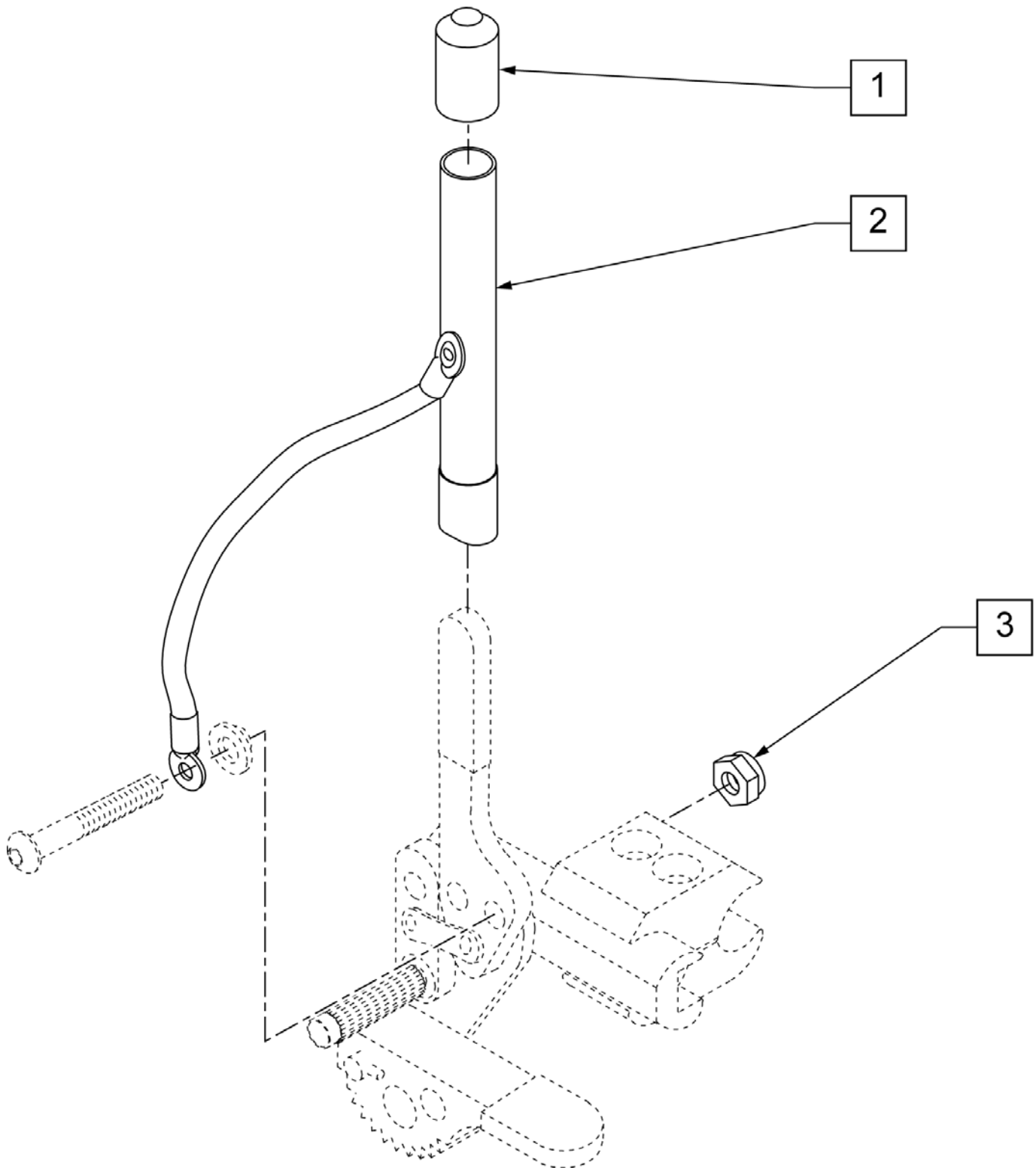


| Pos. | Item Number | Description                   | Remarks |
|------|-------------|-------------------------------|---------|
| 1    | 560600      | 1 1/2 x .705 x .187 FOAM GRIP |         |
| 3    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS   |         |
| 1-3  | 880825      | WHL LOCK 6" EXT TUBE ASSM(EA) |         |
| 1-3  | 920032      | WHEEL LOCK 6" EXT TUBES (PR)  | PAIR    |

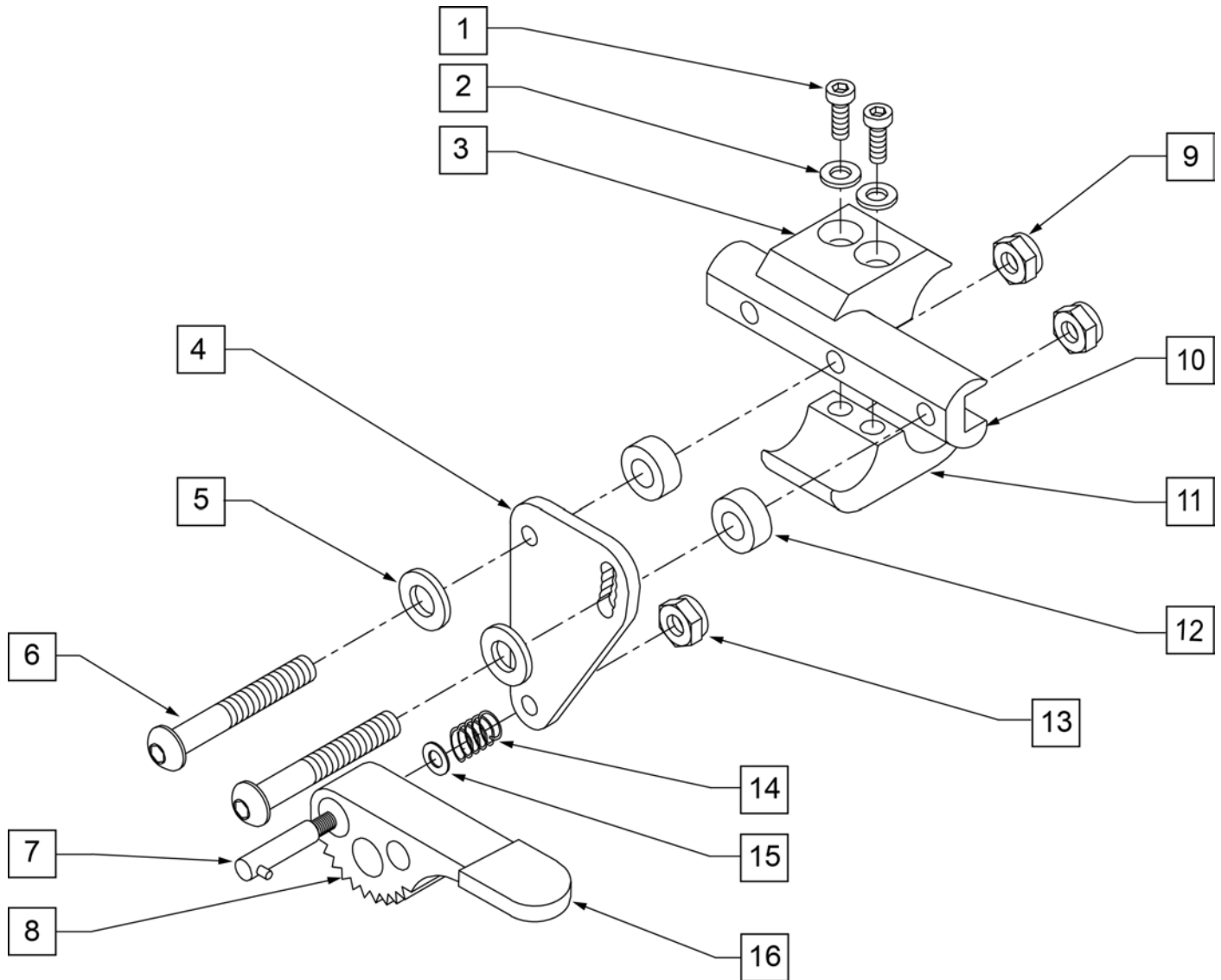


| Pos. | Item Number | Description                         | Remarks |
|------|-------------|-------------------------------------|---------|
| 1    | 107032      | 1/4-28 x 1 SOC C/S BLK              |         |
| 2    | 262114      | 1/4 WASHER HI COLLAR LOCK ZN        |         |
| 3    | 550659      | CLAMP 1 1/4" RECESS HALF BLK        |         |
| 4    | 500274      | GRADE AID PLATE BLK                 |         |
| 5    | 561028      | 1/4" NYLON WASHER                   |         |
| 6    | 226122      | 1/4-28 x 1.10 BH C/S                |         |
| 7    | 481021      | GRADE AID ROD W/PIN WHEEL LOCK      |         |
| 8    | 561090      | GRADE AID WHEEL R NEW               | RIGHT   |
| 8    | 561091      | GRADE AID WHEEL L NEW               | LEFT    |
| 9    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS         |         |
| 10   | 550374      | WHEEL LOCK SLIDE BLOCK 3 1/2" BLACK |         |
| 11   | 550658      | CLAMP 1 1/4" THREADED BLACK         |         |
| 12   | 483027      | 1/4 x 1/4 ALUM SPACER BLK           |         |
| 13   | 106935      | NUT NYLOCK THIN 1/4-28 ROHS         |         |
| 14   | 321130      | GRADE AID SPRING                    |         |
| 15   | 106947      | WASHER FLAT THIN AN 5/16 ROHS       |         |

| Pos.        | Item Number | Description                    | Remarks |
|-------------|-------------|--------------------------------|---------|
| 16          | 560503      | GRADE AID HANDLE COVER         |         |
| 1-3,11      | 107639      | WHL LCK CLMP ASM               |         |
| 4,7,8,13-16 | 880904      | GRADE AID PLATE W/ WHEEL BLACK | RIGHT   |
| 4,7,8,13-16 | 880905      | GRADE AID PLATE W/ WHEEL BLACK | LEFT    |
| 4-10,12-16  | 880912      | GRADE AID ASSM BLACK NO CLAMP  | RIGHT   |
| 4-10,12-16  | 880913      | GRADE AID ASSM BLACK NO CLAMP  | LEFT    |
| 1-16        | 880914      | GRADE AID ASSM BLACK W/ CLAMP  | RIGHT   |
| 1-16        | 880915      | GRADE AID ASSM BLACK W/ CLAMP  | LEFT    |

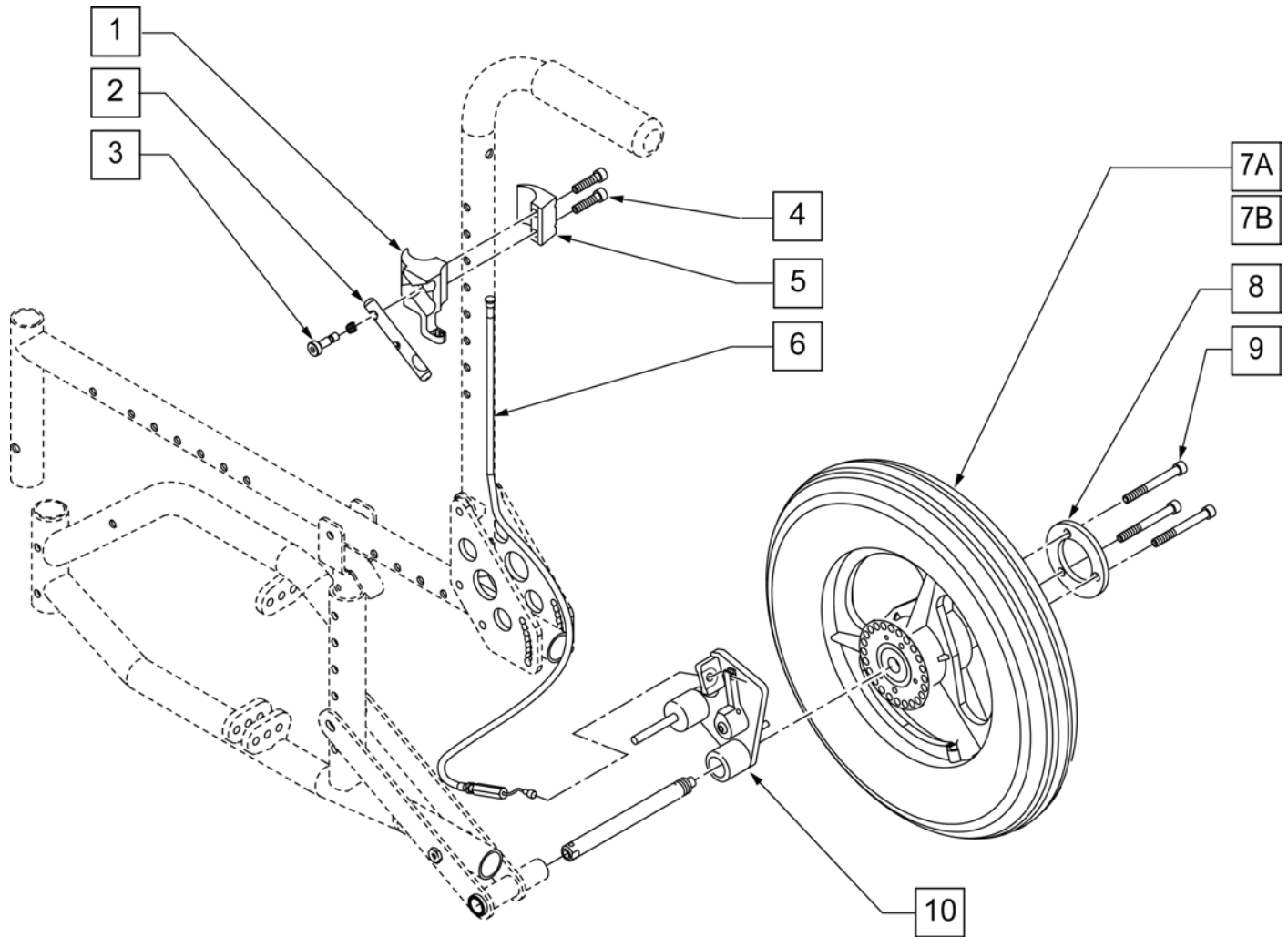

| Pos.      | Item Number | Description                    | Remarks                  |
|-----------|-------------|--------------------------------|--------------------------|
| 1-5       | 922331      | HUB LOCK CLAMP ASSM            | RIGHT                    |
| 1-5       | 922332      | HUB LOCK CLAMP ASSM            | LEFT                     |
| 2         | 506430      | ALUMINUM Z-LOCK TRIGGER        |                          |
| 6         | 922333      | HUB LOCK CABLE ASSM            |                          |
| 7A-9      | 922334      | 12" PNEU W/LOCKING HUB         | PNEUMATIC TIRE           |
| 7B-9      | 922335      | 12" PNEU INSERT W/LOCKING HUB  | PNEUMATIC TIRE W/ INSERT |
| 7C-9      | 105627      | 12" POLY W/ LOCKING HUB        | POLY TIRE                |
| 10        | 922329      | LOWER HUB LOCK ASSM            | RIGHT                    |
| 10        | 922330      | LOWER HUB LOCK ASSM            | LEFT                     |
| 1-7A,8-10 | 922350      | LOCKING HUB ASSM 12" PNEU      | INCLUDES RH & LH ASSMS   |
| 1-7B,8-10 | 922351      | LOCKING HUB ASSM 12"PNEU INSRT | INCLUDES RH & LH ASSMS   |
| 1-7C,8-10 | 105618      | LOCKING HUB ASSM 12" POLY      | INCLUDES RH & LH ASSMS   |


NOTE: QUICKIE STYLE WHEEL LOCKS ARE NOT OFFERED ON THE BREEZY 600 BUT WILL MOUNT DIRECTLY USING THE WHEEL LOCK CLAMP.

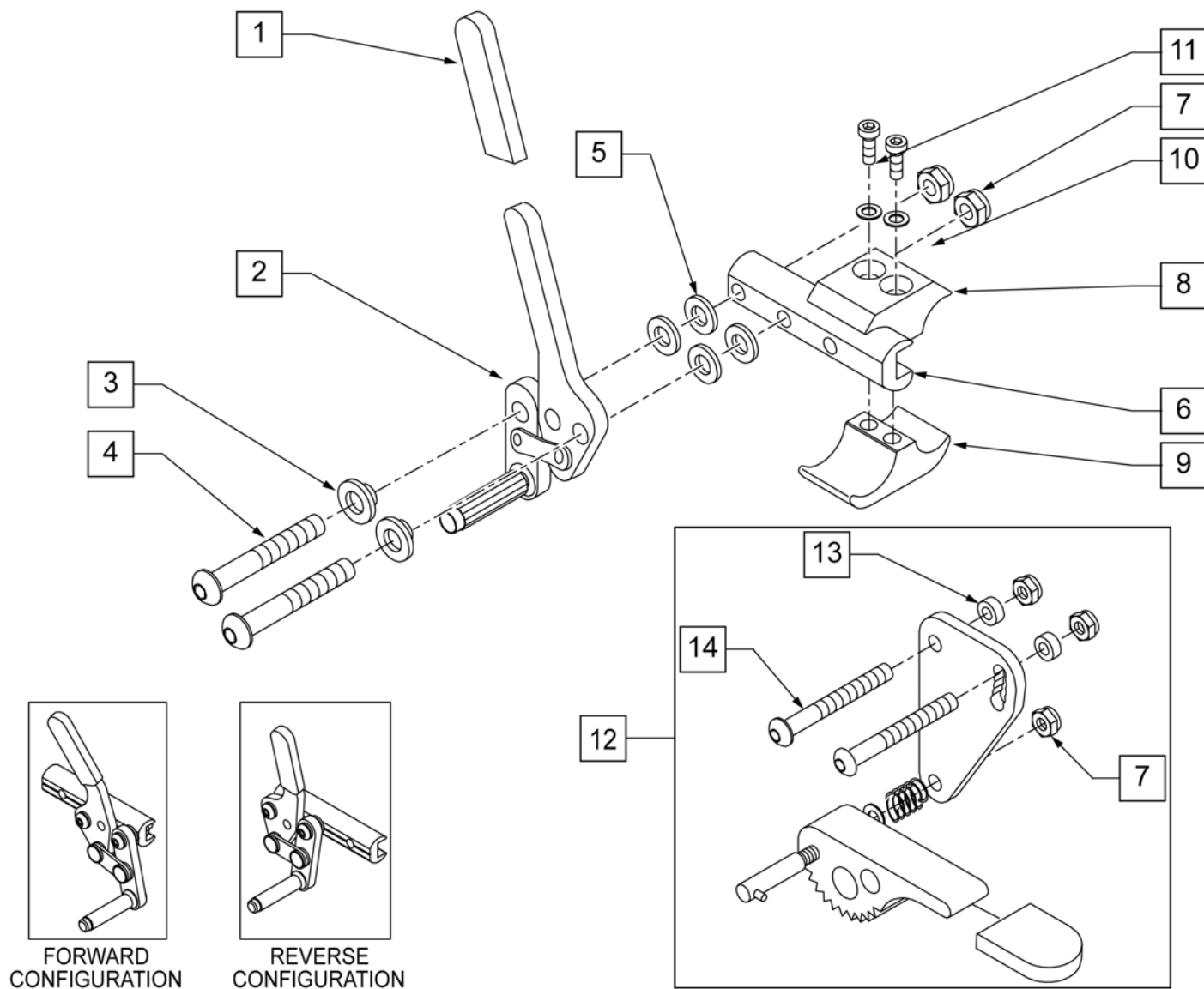

| Pos. | Item Number | Description                       | Remarks |
|------|-------------|-----------------------------------|---------|
| 1    | 560500-QU   | HB PLASTIC COVER                  |         |
| 1,2  | 881748      | WHEEL LOCK ASSM R PUSH            |         |
| 1,2  | 881749      | WHEEL LOCK ASSM L PUSH            |         |
| 3    | 560126      | .250 ID .312 OD BUSHING PLASTIC   |         |
| 4    | 226125      | 1/4-28 X 1 1/8 BH C/S             |         |
| 5    | 561028      | 1/4" NYLON WASHER                 |         |
| 6    | 550374      | WHEEL LOCK SLIDE BLOCK 3 1/2" BLK |         |
| 7    | 106941      | 1/4-28 AUTOMATION NUT             |         |
| 8    | 550659      | CLAMP 1 1/4" RECESS HALF BLK      |         |
| 9    | 550658      | CLAMP 1 1/4" THREADED BLK         |         |

| Pos.    | Item Number | Description                             | Remarks               |
|---------|-------------|-----------------------------------------|-----------------------|
| 10      | 107367      | WASHER LOCK HI COLLAR 1/4 ROHS          |                       |
| 11      | 107032      | SHCS 1/4-28 X 1 ROHS                    |                       |
| 12      | 880916      | GRADE AID PLATE W/WHL & HDWR R          |                       |
| 12      | 880917      | GRADE AID PLATE W/WHL & HDWR L          |                       |
| 6,12    | 880912      | GRADE AID ASSM BLK R N/CLAMP            |                       |
| 6,12    | 880913      | GRADE AID ASSM BLK L N/CLAMP            |                       |
| 13      | 483027      | 1/4 x 1/4 ALUMINUM SPACER BLK           |                       |
| 14      | 226130      | 1/4-28 x 1 1/2 BH C/S                   |                       |
| 8-11    | 107639      | WHL LCK CLMP ASM                        |                       |
| 8-12A   | 880914      | GRADE AID ASSM BLK R W/CLAMP            |                       |
| 8-12A   | 880915      | GRADE AID ASSM BLK L W/CLAMP            |                       |
| 1-7     | 883010      | WHL LOCK PUSH R W/O CLMP BLK            | FORWARD CONFIG.       |
| 1-7     | 883011      | WHL LOCK PUSH L W/O CLMP BLK            | FORWARD CONFIG.       |
| 1-7     | 881762      | WHL LOCK PUSH R REV N/ CLMP BLK         | REVERSE CONFIG.       |
| 1-7     | 881763      | WHL LOCK L PUSH REV N/ CLMP BLK         | REVERSE CONFIG.       |
| 1-11    | 881965      | WHL LOCK PUSH R W/ CLMP BLK             | FORWARD CONFIG.       |
| 1-11    | 881966      | WHL LOCK PUSH L W/ CLMP BLK             | FORWARD CONFIG.       |
| 1-11    | 881973      | WHL LOCK PUSH R REV W/CLMP BLK          | REVERSE CONFIG.       |
| 1-11    | 881974      | WHL LOCK PUSH L REV W/CLMP BLK          | REVERSE CONFIG.       |
| 1-11    | 901551      | WHEEL LOCK PUSH W/CLAMP BLK             | PAIR, FORWARD CONFIG. |
| 1-11    | 901542      | WHEEL LOCK PUSH REV W/CLAMP BLK         | PAIR, REVERSE CONFIG. |
| 1-11,12 | 901557      | WHEEL LOCK PUSH W/GRADE AID W/CLAMP BLK | PAIR                  |
| 1-11,12 | 881969      | WHL LOCK R PUSH W/ G.A. W/CLMP BLK      |                       |
| 1-11,12 | 881970      | WHL LOCK L PUSH W/ G.A. W/CLMP BLK      |                       |
| 1-7,12  | 881754      | WHL LOCK R PUSH W/G.A. N/ CLMP BLK      |                       |
| 1-7,12  | 881755      | WHL LOCK L PUSH W/G.A. N/ CLMP BLK      |                       |


NOTE: QUICKIE STYLE WHEEL LOCKS ARE NOT OFFERED ON THE BREEZY 600 BUT WILL MOUNT DIRECTLY USING THE WHEEL LOCK CLAMP.

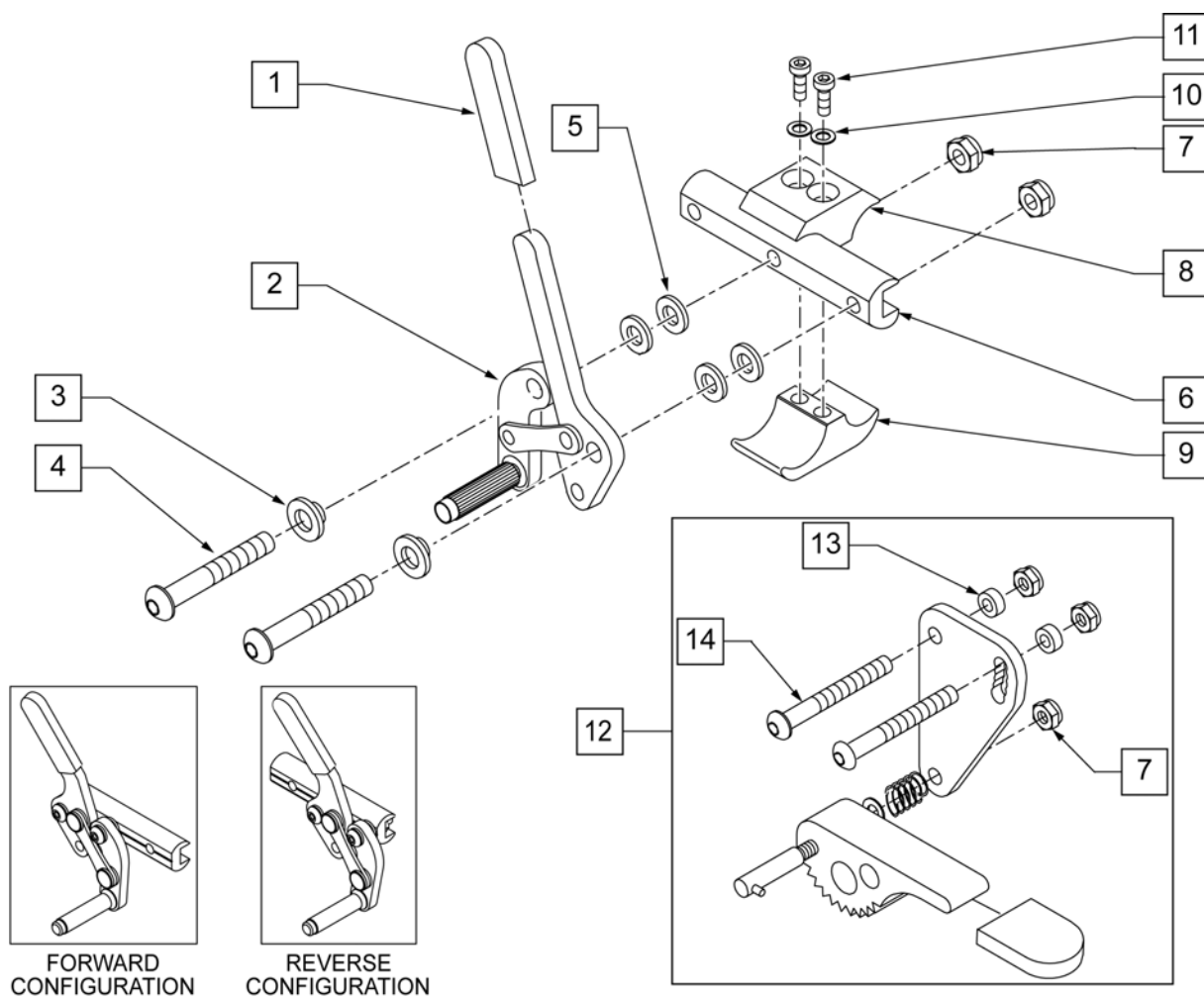


| Pos. | Item Number | Description                       | Remarks |
|------|-------------|-----------------------------------|---------|
| 1    | 560500-QU   | HB PLASTIC COVER                  |         |
| 1,2  | 881798      | WHEEL LOCK ASSM R PULL            |         |
| 1,2  | 881799      | WHEEL LOCK ASSM L PULL            |         |
| 3    | 560126      | .250 ID .312 OD BUSHING PLASTIC   |         |
| 4    | 226125      | 1/4-28 X 1 1/8 BH C/S             |         |
| 5    | 561028      | 1/4" NYLON WASHER                 |         |
| 6    | 550374      | WHEEL LOCK SLIDE BLOCK 3 1/2" BLK |         |
| 7    | 106941      | 1/4-28 AUTOMATION NUT             |         |
| 8    | 550659      | CLAMP 1 1/4" RECESS HALF BLK      |         |
| 9    | 550658      | CLAMP 1 1/4" THREADED BLK         |         |
| 10   | 107367      | WASHER LOCK HI COLLAR 1/4 ROHS    |         |
| 11   | 107032      | SHCS 1/4-28 X 1 ROHS              |         |
| 12   | 880916      | GRADE AID PLATE W/WHL & HDWR R    |         |
| 12   | 880917      | GRADE AID PLATE W/WHL & HDWR L    |         |
| 6,12 | 880912      | GRADE AID ASSM BLK R N/CLAMP      |         |
| 6,12 | 880913      | GRADE AID ASSM BLK L N/CLAMP      |         |



| Pos.    | Item Number | Description                             | Remarks               |
|---------|-------------|-----------------------------------------|-----------------------|
| 13      | 483027      | 1/4 x 1/4 ALUMINUM SPACER BLK           |                       |
| 14      | 226130      | 1/4-28 x 1 1/2 BH C/S                   |                       |
| 8-11    | 107639      | WHL LCK CLMP ASM                        |                       |
| 8-12    | 880914      | GRADE AID ASSM BLK R W/CLAMP            |                       |
| 8-12    | 880915      | GRADE AID ASSM BLK L W/CLAMP            |                       |
| 1-7     | 883012      | WHL LOCK PULL R W/O CLMP BLK            | FORWARD CONFIG.       |
| 1-7     | 883013      | WHL LOCK PULL L W/O CLMP BLK            | FORWARD CONFIG.       |
| 1-7     | 881764      | WHL LOCK PULL R REV N/ CLMP BLK         | REVERSE CONFIG.       |
| 1-7     | 881765      | WHL LOCK L PULL REV N/ CLMP BLK         | REVERSE CONFIG.       |
| 1-11    | 881967      | WHL LOCK PULL R W/ CLMP BLK             | FORWARD CONFIG.       |
| 1-11    | 881968      | WHL LOCK PULL L W/ CLMP BLK             | FORWARD CONFIG.       |
| 1-11    | 881975      | WHL LOCK PULL R REV W/CLMP BLK          | REVERSE CONFIG.       |
| 1-11    | 881976      | WHL LOCK PULL L REV W/CLMP BLK          | REVERSE CONFIG.       |
| 1-11    | 901555      | WHEEL LOCK PULL W/CLAMP BLK             | PAIR, FORWARD CONFIG. |
| 1-11    | 901543      | WHEEL LOCK PULL REV W/CLAMP BLK         | PAIR, REVERSE CONFIG. |
| 1-7,12  | 881756      | WHL LOCK R PULL W/G.A. N/ CLMP BLK      |                       |
| 1-7,12  | 881757      | WHL LOCK L PULL W/G.A. N/ CLMP BLK      |                       |
| 1-11,12 | 881971      | WHL LOCK R PULL W/ G.A. W/CLMP BLK      |                       |
| 1-11,12 | 881972      | WHL LOCK L PULL W/ G.A. W/CLMP BLK      |                       |
| 1-11,12 | 901558      | WHEEL LOCK PULL W/GRADE AID W/CLAMP BLK | PAIR                  |

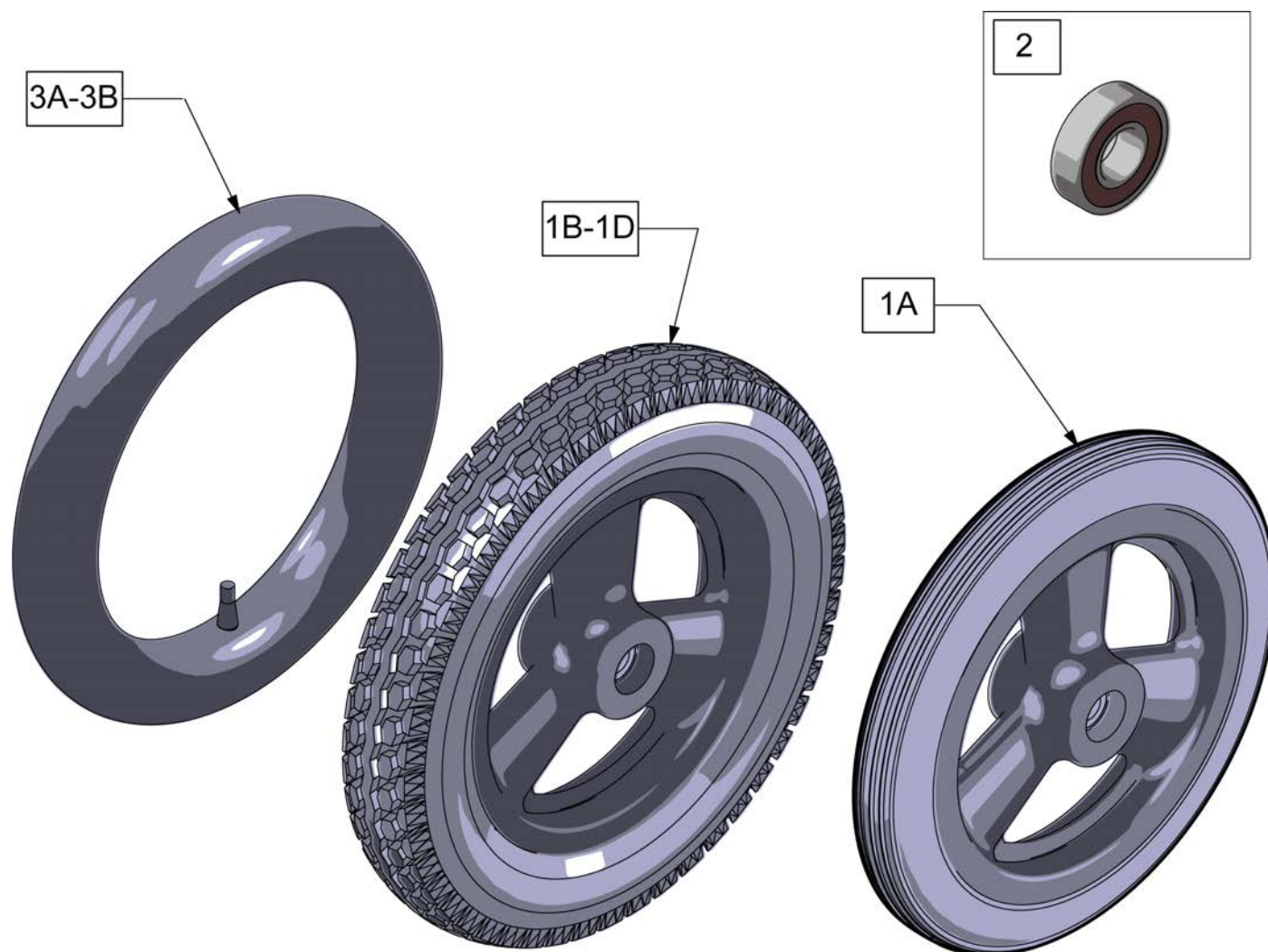

| Pos. | Item Number | Description                 | Remarks   |
|------|-------------|-----------------------------|-----------|
| 1A   | 101466      | 12 IN MAG POLY TIRE 3 SPOKE |           |
| 1B   | 101661      | 12" MAG PNEU 3 SPOKE        |           |
| 1C   | 101662      | 12" MAG PNEU INSRT 3 SPK    |           |
| 1D   | 384706      | TIRE PNEU 12-1/2" X 2-1/4"  | Tire only |
| 2    | 349500      | 1 1/8 X 1/2 BEARING         |           |
| 3A   | 384736      | INNER TUBE 12-1/2" X 2-1/4" |           |
| 3B   | 102972-001  | 12 1/2 IN AIRLESS INSERT    |           |

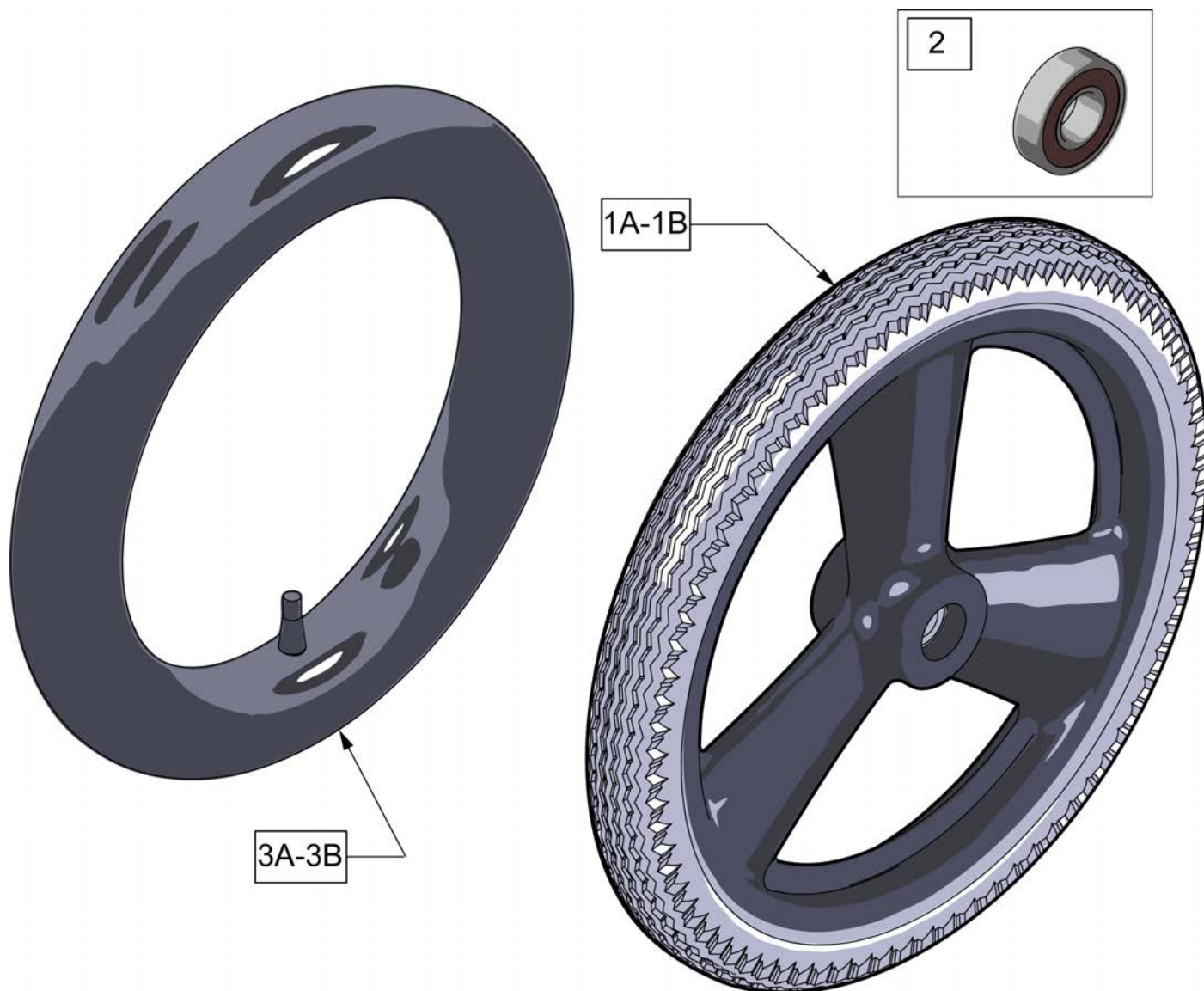

| Pos. | Item Number | Description             | Remarks   |
|------|-------------|-------------------------|-----------|
| 1A   | 101667      | 16 IN MAG PNEU 3 SPOKE  |           |
| 1B   | 101668      | 16 IN MAG PNEU W/INSERT |           |
| 1C   | 385237      | 16" X 1.75" PNEU TIRE   | Tire Only |
| 2    | 349500      | 1 1/8 X 1/2 BEARING     |           |
| 3A   | 384738      | 16" X 1.75" INNER TUBE  |           |
| 3B   | 102973-001  | 16IN AIRLESS INSERT     |           |



| Pos. | Item Number | Description                  | Remarks                                                                 |
|------|-------------|------------------------------|-------------------------------------------------------------------------|
| 1-3  | 401094      | 20" MAG W/BRNG 6 SPOKE       | 20"                                                                     |
| 1-3  | 401093      | 22" MAG W/BRNG 6 SPOKE       | 22"(501mm)                                                              |
| 1-3  | 401092      | 24" MAG W/BRNG 6 SPOKE       | 24"(540mm)                                                              |
| 2    | 349500      | 1 1/8 X 1/2 BEARING          |                                                                         |
| 3    | 521564      | MAG BEARING SPACER           |                                                                         |
| 4    | 101932      | SPCR .194ID X .375 OD X .750 | Used when previously ordered with Omit handrims or Natural Fit Handrims |

**NOTE:** THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.

**NOTE:** THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.

**NOTE:** IF MAG WHEEL ASSEMBLY WAS ORDER WITH OUT HANDRIM AND A HANDRIM IS ADDED TO THE WHEEL QTY 6 OF 561037 AND 281175 MUST BE ORDERED TO PROPERLY MOUNT THE HANDRIM.

**NOTE:** THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.



| Pos. | Item Number | Description                  | Remarks                                 |
|------|-------------|------------------------------|-----------------------------------------|
| 1A   | 116178      | WHEEL ASSY MBL 20"           | 20" (451)                               |
| 1B   | 116179      | WHEEL ASSY MBL 22"           | WHL RIM/SPOKE MBL 22 (501)              |
| 1C   | 116180      | WHEEL ASSY MBL 24"           | WHL RIM/SPOKE MBL 24 (540)              |
| 1D   | 116181      | WHEEL ASSY MBL 25"           | WHL RIM/SPOKE MBL 25 (559)              |
| 1E   | 116182      | WHEEL ASSY MBL 26"           | WHL RIM/SPOKE MBL 26 (590)              |
| 2    | 114097      | BEARING, REPLACEMENT, MBL    | 12.7mm i.d. - 28mm o.d. - 8mm wide      |
| 3A   | 118100      | MBL LT 20IN SPOKE RPLMNT KIT | Kit contains 6 inner and 6 outer spokes |
| 3B   | 118101      | MBL LT 22IN SPOKE RPLMNT KIT | Kit contains 6 inner and 6 outer spokes |
| 3C   | 118102      | MBL LT 24IN SPOKE RPLMNT KIT | Kit contains 6 inner and 6 outer spokes |
| 3D   | 118103      | MBL LT 25IN SPOKE RPLMNT KIT | Kit contains 6 inner and 6 outer spokes |
| 3E   | 118104      | MBL LT 26IN SPOKE RPLMNT KIT | Kit contains 6 inner and 6 outer spokes |

**NOTE:** THIS WHEEL UTILIZES TAB MOUNT HANDRIMS - THE HANDRIM HAS A STRAIGHT TAB THAT MOUNTS TO THE WHEEL THROUGH THE RIM.



| Pos. | Item Number | Description                    | Remarks    |
|------|-------------|--------------------------------|------------|
| 1    | 762438      | SPINERGY STD WHEEL 540 BLK SPK | 24"(540mm) |
| 1    | 762441      | SPINERGY STD WHL 559 BLK SPK   | 25"(559mm) |
| 1    | 762437      | SPINERGY STD WHEEL 590 BLK SPK | 26"(590mm) |
| 1    | 762442      | SPINERGY STD WHEEL 540 BLUE    | 24"(540mm) |
| 1    | 762444      | SPINERGY STD WHEEL 559 BLU     | 25"(559mm) |
| 1    | 762446      | SPINERGY STD WHEEL 590 BLUE    | 26"(590mm) |
| 1    | 762443      | SPINERGY STD WHEEL 540 RED     | 24"(540mm) |
| 1    | 762445      | SPINERGY STD WHEEL 559 RED     | 25"(559mm) |
| 1    | 762447      | SPINERGY STD WHEEL 590 RED     | 26"(590mm) |
| 1    | 762456      | SPINERGY STD WHEEL 540 WHT     | 24"(540mm) |
| 1    | 762463      | SPINERGY STD WHL 559 WHT SPK   | 25"(559mm) |
| 1    | 762467      | SPINERGY STD WHEEL 590 WHT     | 26"(590mm) |
| 1    | 762433      | SPINERGY STD WHEEL 540 YELLOW  | 24"(540mm) |
| 1    | 762462      | SPINERGY STD WHL 559 YLW SPK   | 25"(559mm) |
| 1    | 762431      | SPINERGY STD WHEEL 590 YELLOW  | 26"(590mm) |
| 2    | 349500      | 1 1/8 X 1/2 BEARING            |            |






| Pos. | Item Number | Description                | Remarks                      |
|------|-------------|----------------------------|------------------------------|
| 1    | 384706      | TIRE PNEU 12-1/2" X 2-1/4" | 12-1/2" X 2-1/4"             |
| 1    | 385225      | TIRE PNEUMATIC 18"         | 18" x 1 3/8                  |
| 1    | 385250      | 20" PNEUMATIC TIRE         | 20" x 1 3/8                  |
| 1    | 385275      | 22" PNEUMATIC TIRE         | 22" x 1 3/8 (501mm)          |
| 1    | 385300      | 24" PNEUMATIC TIRE         | 24" x 1 3/8 (540mm)          |
| 1    | 385325      | 26" PNEUMATIC TIRE         | 26" x 1 3/8 (590mm)          |
| 1    | 384614      | 24" x 2.125 PNEUMATIC TIRE | 24"x2 1/8 (540mm) used w/ M6 |





| Pos. | Item Number | Description       | Remarks     |
|------|-------------|-------------------|-------------|
| 1    | 386232      | 20" POLY TIRE STD | 20"         |
| 1    | 386242      | 22" POLY TIRE STD | 22" (501mm) |
| 1    | 386252      | 24" POLY TIRE STD | 24" (540mm) |

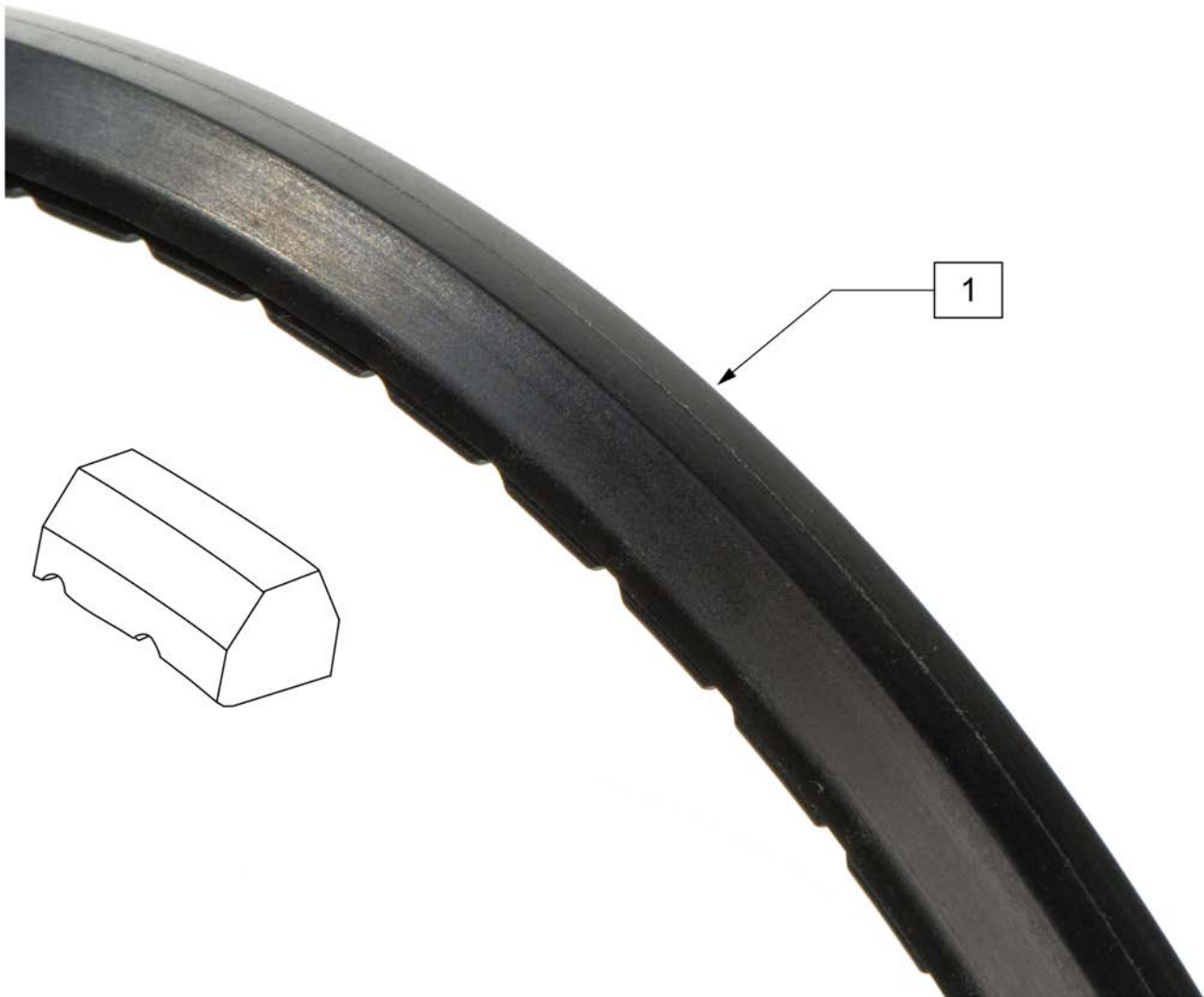


| Pos. | Item Number | Description          | Remarks    |
|------|-------------|----------------------|------------|
| 1    | 386262      | 20" LP POLY TIRE BR  | 20"        |
| 1    | 386263      | 22" LP POLY TIRE BR  | 22"(501mm) |
| 1    | 386254      | 24" POLY LOW PROFILE | 24"(540mm) |



| Pos. | Item Number | Description                         | Remarks     |
|------|-------------|-------------------------------------|-------------|
| 1    | 385280      | 22" x 1" HI PRES CLINCHER           | 22" (501mm) |
| 1    | 389020      | 24" x 1" HI PRES CLNCHR TIRE        | 24" (540mm) |
| 1    | 389012      | QP 24" x 1" HT TURB BLK WR/BD 559   | 25" (559mm) |
| 1    | 389030      | 26" x 1" HI PRES CLNCHR TIRE        | 26" (590mm) |
| 1    | 105919      | 22" x 1" HI PRESS CLNCHR ULTRALIGHT | 22" (489mm) |




| Pos. | Item Number | Description     |
|------|-------------|-----------------|
| 1    | 385350      | 24" KEVLAR TIRE |

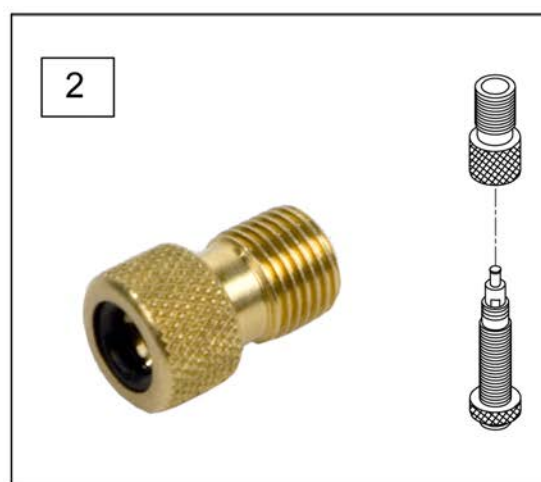

| Pos. | Item Number | Description                   | Remarks           |
|------|-------------|-------------------------------|-------------------|
| 1A   | 382305      | 22" X 1" TUBE PRESTA VALVE    |                   |
| 1A   | 382330      | 24" TUBE PRESTA VALVE (540MM) |                   |
| 1A   | 384545      | QP 24" HT INNER TUBE (559MM)  |                   |
| 1A   | 382355      | 26" TUBE PRESTA VALVE (590MM) |                   |
| 1A   | 384550      | QP 26" HT INNER TUBE (700C)   |                   |
| 1B   | 384736      | INNER TUBE 12-1/2" X 2-1/4"   |                   |
| 1B   | 384738      | 16" X 1.75" INNER TUBE        |                   |
| 1B   | 382255      | TUBE 18" TIRE                 | 18" X 1-3/4" TUBE |
| 1B   | 382275      | 20" TUBE                      | 20" 1-3/8" TUBE   |
| 1B   | 382300      | 22" TUBE                      | 22" X 1-3/8" TUBE |
| 1B   | 382336      | TUBE PRIMO 24" 25-540 SV      | 24" X 1" TUBE     |
| 1B   | 382325      | 24 X 1.3/8IN TUBE REG         |                   |
| 1B   | 382334      | TUBE INLINE 24 X 1.75         |                   |
| 1B   | 384551      | 26" X 1" TUBE SCHRADER        |                   |
| 1B   | 382350      | 26" TUBE                      | 26" X 1-3/8" TUBE |
| 1B   | 382361      | TUBE 26 X 1.5/1.95/ S/V       |                   |
| 1B   | 382365      | 26 X 1.25/1.5 TUBE SHRADER    |                   |
| 1B   | 382375      | 24" x 2.125 TUBE              |                   |
| 1C   | 102972-001  | 12 1/2 IN AIRLESS INSERT      | 12-1/2" X 2-1/4"  |
| 1C   | 102973-001  | 16IN AIRLESS INSERT           | 16" X 1-3/4"      |
| 1C   | 102973-002  | 18IN AIRLESS INSERT           | 18" X 1-3/4"      |


| Pos. | Item Number | Description                  | Remarks      |
|------|-------------|------------------------------|--------------|
| 1C   | 102973-003  | 20IN AIRLESS INSERT          | 20" X 1-3/8" |
| 1C   | 102972-002  | AIRLESS INSERT 20 X 2-1/8 P3 |              |
| 1C   | 102973-004  | 22IN AIRLESS INSERT          | 22" X 1-3/8" |
| 1C   | 102973-005  | 24IN AIRLESS INSERT          | 24" X 1-3/8" |
| 1C   | 102972-003  | AIRLESS INSERT 24 X 2-1/8    |              |
| 1C   | 108259      | 26" AIRLESS INSERT           | 26" X 1-3/8" |
| 2    | 381230      | VALVE ADAPTER PRESTA         |              |

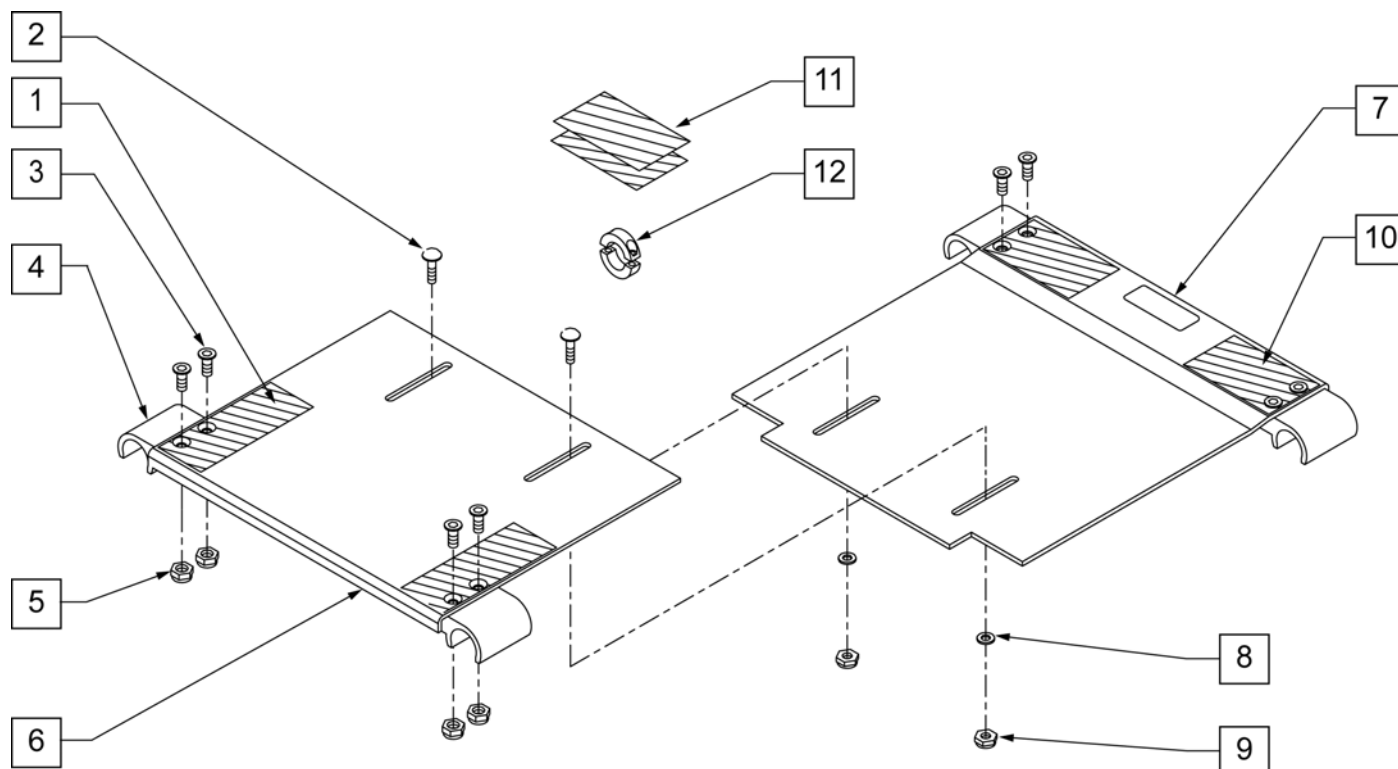

| Pos. | Item Number | Description                         | Remarks                                 |
|------|-------------|-------------------------------------|-----------------------------------------|
| 1    | 751047      | HOOK FASTENER ZGS FRONT             |                                         |
| 2    | 455036-QU   | 1/4-28 x 9/16 SPEC ROUND HD         |                                         |
| 3    | 113278      | FHCS 1/4 -28 x 3/4 ROHS             |                                         |
| 4    | 560112      | SEAT SNAP ZGS                       |                                         |
| 5    | 106934      | NUT NYLOCK 1/4 -28 ROHS             |                                         |
| 6    | 124340      | SEAT PAN SMALL 10" x 10" ZGS        | FITS ZGS ONLY FITS 10" -12" SEAT DEPTHS |
| 6    | 124341      | SEAT PAN SMALL 10" x 12" ZGS        | FITS 10" -12" SEAT DEPTHS               |
| 6    | 124342      | SEAT PAN SMALL 10" x 14" ZGS        | FITS 10" -12" SEAT DEPTHS               |
| 6    | 124359      | SEAT PAN SMALL 10 x 16" ZGS         | FITS 10" -12" SEAT DEPTHS               |
| 6    | 124343      | SEAT PAN MED 12" x 10" ZGS          | FITS ZGS ONLY FITS 12" -16" SEAT DEPTHS |
| 6    | 124344      | SEAT PAN MED 12" x 12" ZGS          | FITS 12" -16" SEAT DEPTHS               |
| 6    | 124345      | SEAT PAN MED 12" x 14" ZGS          | FITS 12" -16" SEAT DEPTHS               |
| 6    | 124360      | SEAT PAN MED 12 x 16" ZGS           | FITS 12" -16" SEAT DEPTHS               |
| 6    | 124346      | SEAT PAN LARGE 14" x 10" ZGS        | FITS ZGS ONLY FITS 14" -18" SEAT DEPTHS |
| 6    | 124347      | SEAT PAN LARGE 14" x 12" ZGS        | FITS 14" -18" SEAT DEPTHS               |
| 6    | 124348      | SEAT PAN LARGE 14" x 14" ZGS        | FITS 14" -18" SEAT DEPTHS               |
| 6    | 124361      | SEAT PAN LARGE 14 X 16" ZGS         | FITS 14" -18" SEAT DEPTHS               |
| 6    | 124362      | SEAT PAN X-LARGE 16 x 10" ZGS       | FITS ZGS ONLY FITS 16" -20" DEPTHS      |
| 6    | 124363      | SEAT PAN X-LARGE 16 x 12" ZGS       | FITS 16" -20" SEAT DEPTHS               |
| 6    | 124364      | SEAT PAN X-LARGE 16 x 14" ZGS       | FITS 16" -20" SEAT DEPTHS               |
| 6    | 124365      | SEAT PAN X-LARGE 16 x 16" ZGS       | FITS 16" -20" SEAT DEPTHS               |
| 7    | 124350      | SEAT PAN SMALL OFFSET 10" x 10" ZGS | FITS ZGS ONLY FITS 10" -12" SEAT DEPTHS |
| 7    | 124351      | SEAT PAN SMALL OFFSET 10" x 12" ZGS | FITS 10" -12" SEAT DEPTHS               |
| 7    | 124352      | SEAT PAN SMALL OFFSET 10" x 14" ZGS | FITS 10" -12" SEAT DEPTHS               |
| 7    | 124370      | SEAT PAN OFFSET SMALL 10 x 16" ZGS  | FITS 10" -12" SEAT DEPTHS               |
| 7    | 124353      | SEAT PAN MED OFFSET 12" x 10" ZGS   | FITS ZGS ONLY FITS 14" -18" SEAT DEPTHS |
| 7    | 124354      | SEAT PAN MED OFFSET 12" x 12" ZGS   | FITS 14" -18" SEAT DEPTHS               |
| 7    | 124355      | SEAT PAN MED OFFSET 12" x 14" ZGS   | FITS 14" -18" SEAT DEPTHS               |

| Pos. | Item Number | Description                          | Remarks                                 |
|------|-------------|--------------------------------------|-----------------------------------------|
| 7    | 124371      | SEAT PAN OFFSET MED 12 x 16" ZGS     | FITS 14" -18" SEAT DEPTHS               |
| 7    | 124356      | SEAT PAN LARGE OFFSET 14" x 10" ZGS  | FITS ZGS ONLY FITS 14" -18" SEAT DEPTHS |
| 7    | 124357      | SEAT PAN LARGE OFFSET 14" x 12" ZGS  | FITS 14" -18" SEAT DEPTHS               |
| 7    | 124358      | SEAT PAN LARGE OFFSET 14" x 14" ZGS  | FITS 14" -18" SEAT DEPTHS               |
| 7    | 124372      | SEAT PAN OFFSET LARGE 14 X 16" ZGS   | FITS 14" -18" SEAT DEPTHS               |
| 7    | 124373      | SEAT PAN OFFSET X-LARGE 16 x 10" ZGS | FITS ZGS ONLY FITS 16" -20" DEPTHS      |
| 7    | 124374      | SEAT PAN OFFSET X-LARGE 16 x 12" ZGS | FITS 16" -20" SEAT DEPTHS               |
| 7    | 124375      | SEAT PAN OFFSET X-LARGE 16 x 14" ZGS | FITS 16" -20" SEAT DEPTHS               |
| 7    | 124376      | SEAT PAN OFFSET X-LARGE 16 x 16" ZGS | FITS 16" -20" SEAT DEPTHS               |
| 8    | 106943      | WASHER FLAT AN 1/4 ROHS              |                                         |
| 9    | 106935      | NUT NYLOCK THIN 1/4 -28 ROHS         |                                         |
| 10   | 751048      | HOOK FASTENER ZGS REAR               |                                         |
| 11   | 751049      | HOOK FASTENER ZGS FLOATING           |                                         |
| 12   | 342048      | ADJ SEAT STOP 1"                     |                                         |
| 1-12 | 703200      | SEAT PAN ASSM SHORT 10-12"W ZGS      | FITS ZGS ONLY FITS 10" -12" SEAT DEPTHS |
| 1-12 | 703201      | SEAT PAN ASSM SHORT 12-14"W ZGS      | FITS 10" -12" SEAT DEPTHS               |
| 1-12 | 703202      | SEAT PAN ASSM SHORT 14-16"W ZGS      | FITS 10" -12" SEAT DEPTHS               |
| 1-12 | 703203      | SEAT PAN ASSM SHORT 16-18"W ZGS      | FITS 10" -12" SEAT DEPTHS               |
| 1-12 | 703210      | SEAT PAN ASSM REG 10-12"W ZGS        | FITS ZGS ONLY FITS 12" -16" SEAT DEPTHS |
| 1-12 | 703211      | SEAT PAN ASSM REG 12-14"W ZGS        | FITS 12" -16" SEAT DEPTHS               |
| 1-12 | 703212      | SEAT PAN ASSM REG 14-16"W ZGS        | FITS 12" -16" SEAT DEPTHS               |
| 1-12 | 703213      | SEAT PAN ASSM REG 16-18"W ZGS        | FITS 12" -16" SEAT DEPTHS               |
| 1-12 | 703220      | SEAT PAN ASSM LONG 10-12"W ZGS       | FITS ZGS ONLY FITS 14" -18" SEAT DEPTHS |
| 1-12 | 703221      | SEAT PAN ASSM LONG 12-14"W ZGS       | FITS 14" -18" SEAT DEPTHS               |
| 1-12 | 703222      | SEAT PAN ASSM LONG 14-16"W ZGS       | FITS 14" -18" SEAT DEPTHS               |
| 1-12 | 703223      | SEAT PAN ASSM LONG 16-18"W ZGS       | FITS 14" -18" SEAT DEPTHS               |
| 1-12 | 703230      | SEAT PAN ASSM X-LONG 10-12"W ZGS     | FITS ZGS ONLY FITS 16" -20" DEPTHS      |
| 1-12 | 703231      | SEAT PAN ASSM X-LONG 12-14"W ZGS     | FITS 16" -20" SEAT DEPTHS               |
| 1-12 | 703232      | SEAT PAN ASSM X-LONG 14-16"W ZGS     | FITS 16" -20" SEAT DEPTHS               |
| 1-12 | 703233      | SEAT PAN ASSM X-LONG 16-18"W ZGS     | FITS 16" -20" SEAT DEPTHS               |



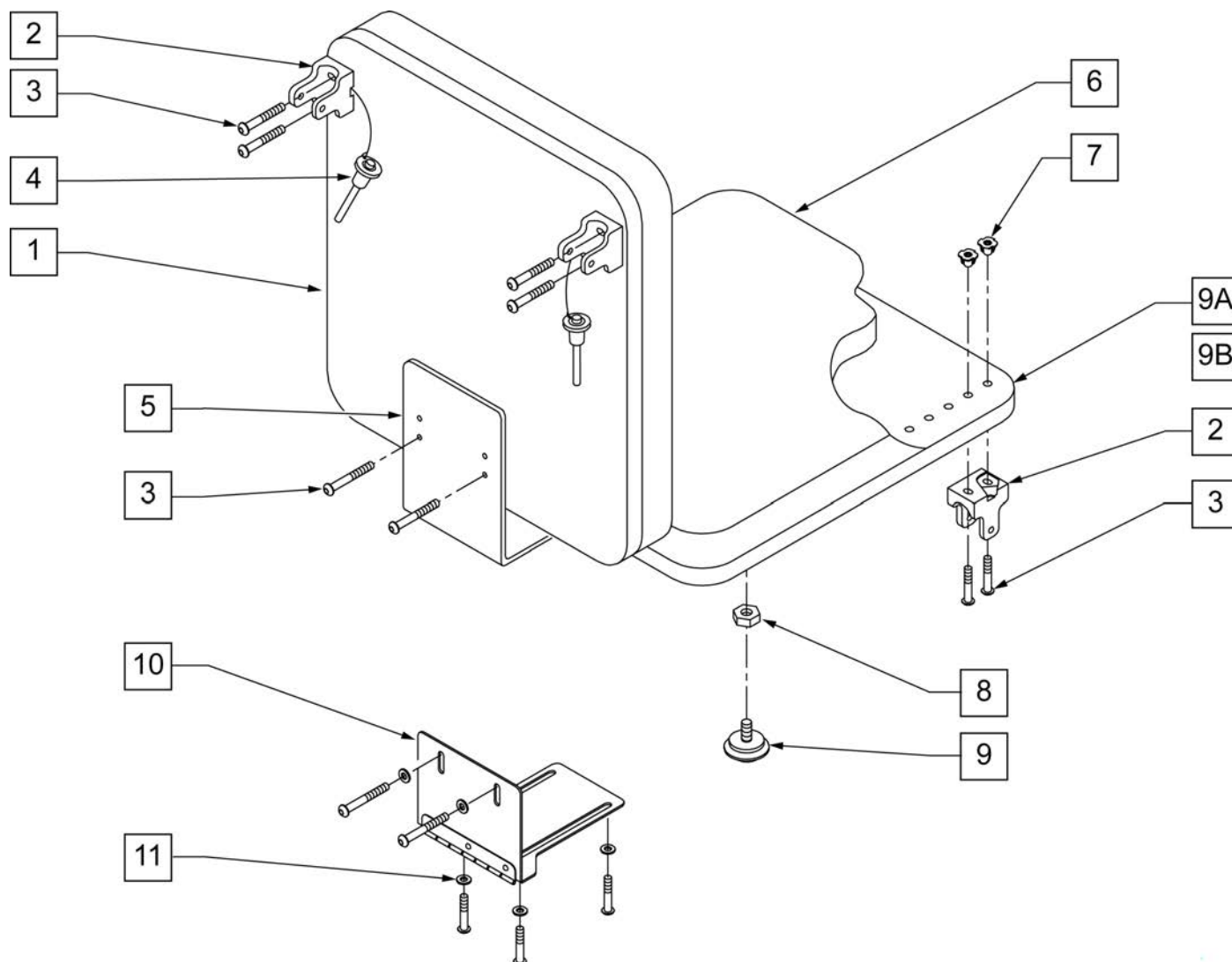


| Pos. | Item Number | Description               | Remarks |
|------|-------------|---------------------------|---------|
| 1,7  | 735050S     | PLY BACK 10" LOW ZP's     |         |
| 1,7  | 735051S     | PLY BACK 10" MED ZP's     |         |
| 1,7  | 735052S     | PLY BACK 10" TALL ZP's    |         |
| 1,7  | 735053S     | PLY BACK 11" LOW ZP's     |         |
| 1,7  | 735054S     | PLY BACK 11" MED ZP's     |         |
| 1,7  | 735055S     | PLY BACK 11" TALL ZP's    |         |
| 1,7  | 735056S     | PLY BACK 12" LOW LOW ZPS  |         |
| 1,7  | 735057S     | PLY BACK 12" MED LOW ZPS  |         |
| 1,7  | 735058S     | PLY BACK 12" TALL LOW ZPS |         |
| 1,7  | 735059S     | PLY BACK 13" LOW LOW ZPS  |         |
| 1,7  | 735060S     | PLY BACK 13" MED LOW ZPS  |         |
| 1,7  | 735061S     | PLY BACK 13" TALL LOW ZPS |         |
| 1,7  | 735062S     | PLY BACK 14" LOW LOW ZPS  |         |
| 1,7  | 735063S     | PLY BACK 14" MED LOW ZPS  |         |
| 1,7  | 735064S     | PLY BACK 14" TALL LOW ZPS |         |
| 1,7  | 735065S     | PLY BACK 15" LOW LOW ZPS  |         |
| 1,7  | 735066S     | PLY BACK 15" MED LOW ZPS  |         |
| 1,7  | 735067S     | PLY BACK 15" TALL LOW ZPS |         |


| Pos.       | Item Number | Description                   | Remarks        |
|------------|-------------|-------------------------------|----------------|
| 1,7        | 735068S     | PLY BACK 16" LOW LOW ZPS      |                |
| 1,7        | 735069S     | PLY BACK 16" MED LOW ZPS      |                |
| 1,7        | 735070S     | PLY BACK 16" TALL LOW ZPS     |                |
| 2          | 562609      | BRACKET SEAT QTS/BACK Z2      |                |
| 3          | 108107      | BHCS 1/4-20 x 3/4 ROHS        |                |
| 4          | 349020      | STROLLER HANDLE RELEASE PIN   |                |
| 5          | 505253      | QT SEAT/BACK BRACKET NEW      |                |
| 7          | 2400        | 1/4-20 X 5/16" T-NUTS 3 PRONG |                |
| 6,7        | 735000S     | PLY SEAT 10 x 10/11 x 1/2 ZP  |                |
| 6,7        | 735001S     | PLY SEAT 10 x 11/12 x 1/2 ZP  |                |
| 6,7        | 735002S     | PLY SEAT 10 x 12/13 x 1/2 ZP  |                |
| 6,7        | 735003S     | PLY SEAT 10 x 13/14 x 1/2 ZP  |                |
| 6,7        | 735004S     | PLY SEAT 10 x 14/15 x 1/2 ZP  |                |
| 6,7        | 735005S     | PLY SEAT 10 x 15/16 x 1/2 ZP  |                |
| 6,7        | 735006S     | PLY SEAT 11 x 10/11 x 1/2 ZP  |                |
| 6,7        | 735007S     | PLY SEAT 11 x 11/12 x 1/2 ZP  |                |
| 6,7        | 735008S     | PLY SEAT 11 x 12/13 x 1/2 ZP  |                |
| 6,7        | 735009S     | PLY SEAT 11 x 13/14 x 1/2 ZP  |                |
| 6,7        | 735010S     | PLY SEAT 11 x 14/15 x 1/2 ZP  |                |
| 6,7        | 735011S     | PLY SEAT 11 x 15/16 x 1/2 ZP  |                |
| 6,7        | 735012S     | PLY SEAT 12 x 10/11 x 1/2 ZP  |                |
| 6,7        | 735013S     | PLY SEAT 12 x 11/12 x 1/2 ZP  |                |
| 6,7        | 735014S     | PLY SEAT 12 x 12/13 x 1/2 ZP  |                |
| 6,7        | 735015S     | PLY SEAT 12 x 13/14 x 1/2 ZP  |                |
| 6,7        | 735016S     | PLY SEAT 12 x 14/15 x 1/2 ZP  |                |
| 6,7        | 735017S     | PLY SEAT 12 x 15/16 x 1/2 ZP  |                |
| 6,7        | 735018S     | PLY SEAT 13 x 10/11 x 1/2 ZP  |                |
| 6,7        | 735019S     | PLY SEAT 13 x 11/12 x 1/2 ZP  |                |
| 6,7        | 735020S     | PLY SEAT 13 x 12/13 x 1/2 ZP  |                |
| 6,7        | 735021S     | PLY SEAT 13 x 13/14 x 1/2 ZP  |                |
| 6,7        | 735022S     | PLY SEAT 13 x 14/15 x 1/2 ZP  |                |
| 6,7        | 735023S     | PLY SEAT 13 x 15/16 x 1/2 ZP  |                |
| 6,7        | 735024S     | PLY SEAT 14 x 10/11 x 1/2 ZP  |                |
| 6,7        | 735025S     | PLY SEAT 14 x 11/12 x 1/2 ZP  |                |
| 6,7        | 735026S     | PLY SEAT 14 x 12/13 x 1/2 ZP  |                |
| 6,7        | 735027S     | PLY SEAT 14 x 13/14 x 1/2 ZP  |                |
| 6,7        | 735028S     | PLY SEAT 14 x 14/15 x 1/2 ZP  |                |
| 6,7        | 735029S     | PLY SEAT 14 x 15/16 x 1/2 ZP  |                |
| 6,7        | 735030S     | PLY SEAT 15 x 10/11 x 1/2 ZP  |                |
| 6,7        | 735031S     | PLY SEAT 15 x 11/12 x 1/2 ZP  |                |
| 6,7        | 735032S     | PLY SEAT 15 x 12/13 x 1/2 ZP  |                |
| 6,7        | 735033S     | PLY SEAT 15 x 13/14 x 1/2 ZP  |                |
| 6,7        | 735034S     | PLY SEAT 15 x 14/15 x 1/2 ZP  |                |
| 6,7        | 735035S     | PLY SEAT 15 x 15/16 x 1/2 ZP  |                |
| 6,7        | 735036S     | PLY SEAT 16 x 10/11 x 1/2 ZP  |                |
| 6,7        | 735037S     | PLY SEAT 16 x 11/12 x 1/2 ZP  |                |
| 6,7        | 735038S     | PLY SEAT 16 x 12/13 x 1/2 ZP  |                |
| 6,7        | 735039S     | PLY SEAT 16 x 13/14 x 1/2 ZP  |                |
| 6,7        | 735040S     | PLY SEAT 16 x 14/15 x 1/2 ZP  |                |
| 6,7        | 735041S     | PLY SEAT 16 x 15/16 x 1/2 ZP  |                |
| 8          | 113281      | NUT HEX THIN 1/4 -20 ROHS     |                |
| 9          | 349560      | ZP RUBBER STOP                |                |
| 10         | 450990      | SEAT TO BACK BRKT 90D CV      | HINGED BRACKET |
| 11         | 106943      | WASHER FLAT AN 1/4 ROHS       |                |
| 2-4,8,9    | 920451      | ST/BACK HDWR KT STD BRKT ZP   |                |
| 2-4,8,9,11 | 920452      | ST/BCK HDWR KIT HNG BRKT ZP   |                |



**NOTE:** HOLE ON TOP OF STROLLER HANDLE (FOR STROLLER HANDLE EXTENSION) EFFECTIVE 12/21/93. BEGINNING SERIAL NUMBER ZP-32105.

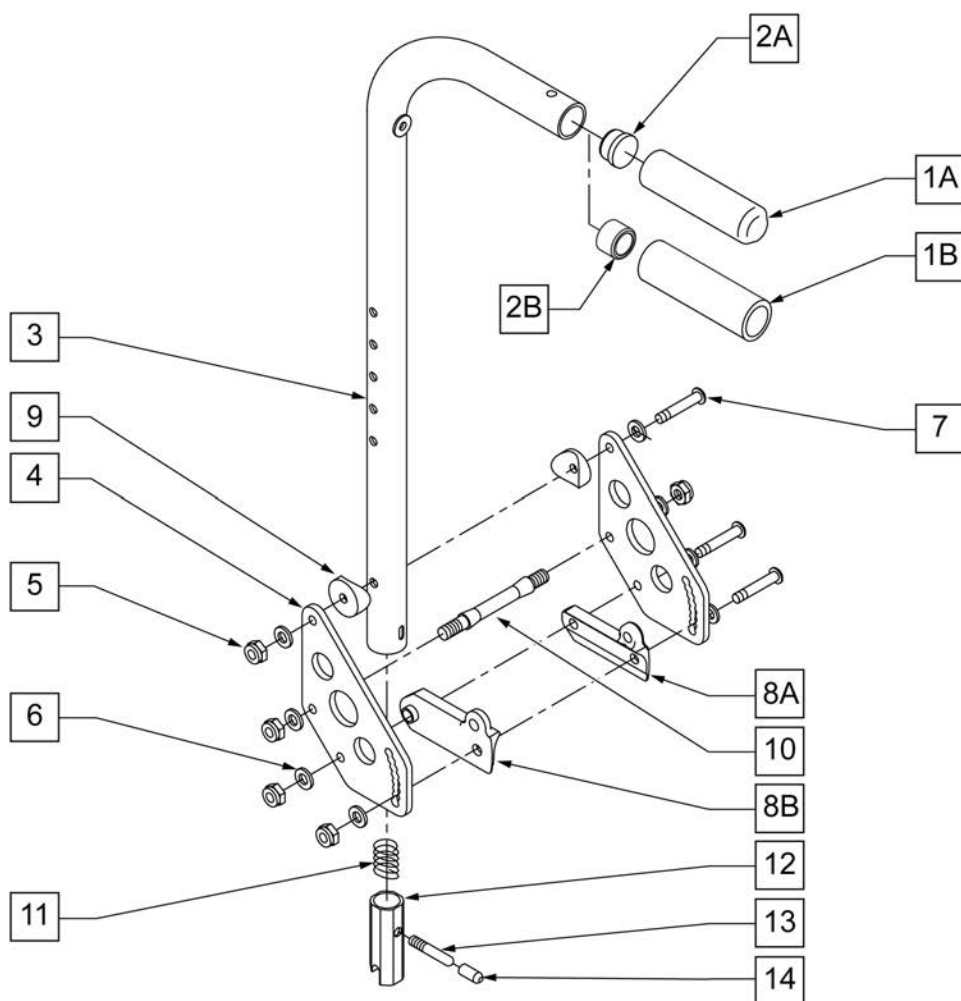

#### MEASURING GUIDE

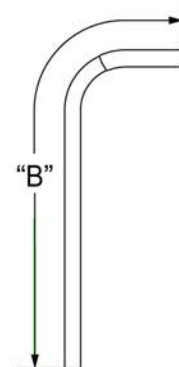


| BACK TUBE | "A"     |
|-----------|---------|
| STD/LOW   | 25 3/4" |
| MED       | 28 3/4" |
| TALL      | 31 3/4" |

**NOTE:** ABOVE DIMENSIONS ARE FOR BACK TUBE ONLY AND DO NOT INCLUDE FOAM GRIP.



| Pos. | Item Number | Description                    | Remarks |
|------|-------------|--------------------------------|---------|
| 1A   | 920027      | QUICKIE GRIP 1" W/INSTR SHEET  |         |
| 1B   | 920023      | QUICKIE GRIP 1" W/HOLE W/INSTR |         |
| 2A   | 561002      | 1" INSERT PLUG .083            |         |
| 2B   | 561082      | 1" TUBE END BUSHING THIN       |         |
| 4    | 506136      | BACKPLATE DAB                  |         |
| 5    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS    |         |
| 6    | 106943      | 1/4 AN WASHER FLAT             |         |
| 7    | 106964      | 1/4-28 x 2 BH C/S              |         |
| 8    | 560518      | SADDLE BACKREST Q2 DAB         | RIGHT   |
| 8    | 560519      | SADDLE BACKREST Q2 DAB         | LEFT    |
| 9    | 561004      | 1" SADDLE 1/4" HOLE            |         |
| 10   | 481042      | BACK RELEASE STOP PIN QTS      |         |
| 11   | 321110      | S/A LATCH SPRING               |         |
| 12   | 483019      | BACK RELEASE LATCH             |         |

| Pos.          | Item Number | Description                      | Remarks |
|---------------|-------------|----------------------------------|---------|
| 13            | 481084      | BACK RELEASE PIN Z2/QT/EX        |         |
| 14            | 560502      | ANTI-TIP LEVER COVER             |         |
| 5-8,10        | 920539      | BACKPLATE DAB HDWR KIT           |         |
| 5-9           | 920921      | BACK PLATE HARDWARE KIT ZTS      |         |
| 5-8,10-14     | 920328      | BACKREST LATCH KIT               |         |
| 1A,2A,3       | 842079      | STRL HANDLE W/FOAM MED ZPS/QT    | Med     |
| 1A,2A,3       | 842080      | STRL HNDL W/FOAM STD ZPS/P5/QT   | Std/Low |
| 1A,2A,3       | 842081      | STRL HANDLE W/FOAM TALL ZPS/QT   | Tall    |
| 1A,2A,3,11-14 | 920389      | STRL HNDL W/LTCH MED ZPS/P5/QTS  | Med     |
| 1A,2A,3,11-14 | 920390      | STRL HANDLE W/LATCH ZPS/P5/QT    | Std/Low |
| 1A,2A,3,11-14 | 920391      | STRL HNDL W/LATCH TALL ZPS/P5/QT | Tall    |
| 1A,2A-14      | 881845      | STRL HNDL MED&PLT ASSM DAB       | Med     |
| 1A,2A-14      | 881846      | STRL HNDL STD&PLT ASSM DAB       | Std/Low |
| 1A,2A-14      | 881847      | STRL HNDL TLL&PLT ASSM DAB       | Tall    |
| 1A,2A-14      | 920999      | STRL HNDL M AND PLTE ASSM ZTS    | Med     |
| 1A,2A-14      | 920945      | STRL HNDL STD AND PLT ASSM ZTS   | Std/Low |
| 1A,2A-14      | 920947      | STRL HNDL T AND PLT ASSM ZTS     | Tall    |
| 1B,2B,3,14    | 842066      | EXT STROLLER HANDLE MED          | Med     |
| 1B,2B,3,14    | 842067      | EXT STROLLER HANDLE STD          | Std/Low |
| 1B,2B,3,14    | 842068      | EXT STROLLER HANDLE TALL         | Tall    |
| 1B,2B,3,11-14 | 920423      | EXT STROLLER HNDL MED W/LATCH    | Med     |
| 1B,2B,3,11-14 | 920421      | EXT STROLLER HNDL STD W/LATCH    | Std/Low |
| 1B,2B,3,11-14 | 920424      | EXT STROLLER HNDL TALL W/LATCH   | Tall    |
| 1B,2B,3-14    | 920761      | EXT STRL HNDL MED&PLT ASSM DAB   | Med     |
| 1B,2B,3-14    | 920762      | EXT STRL HNDL STD&PLT ASSM DAB   | Std/Low |
| 1B,2B,3-14    | 920763      | EXT STRL HNDL TLL&PLT ASSM DAB   | Tall    |
| 1B,2B,3-14    | 920981      | EXT STRLR HNDL M PLST ASSM ZTS   | Med     |
| 1B,2B,3-14    | 920948      | EXT STRLR HNDL STD PLT ASM ZTS   | Std/Low |
| 1B,2B,3-14    | 920949      | EXT STRLR HNDL T PLT ASSM ZTS    | Tall    |

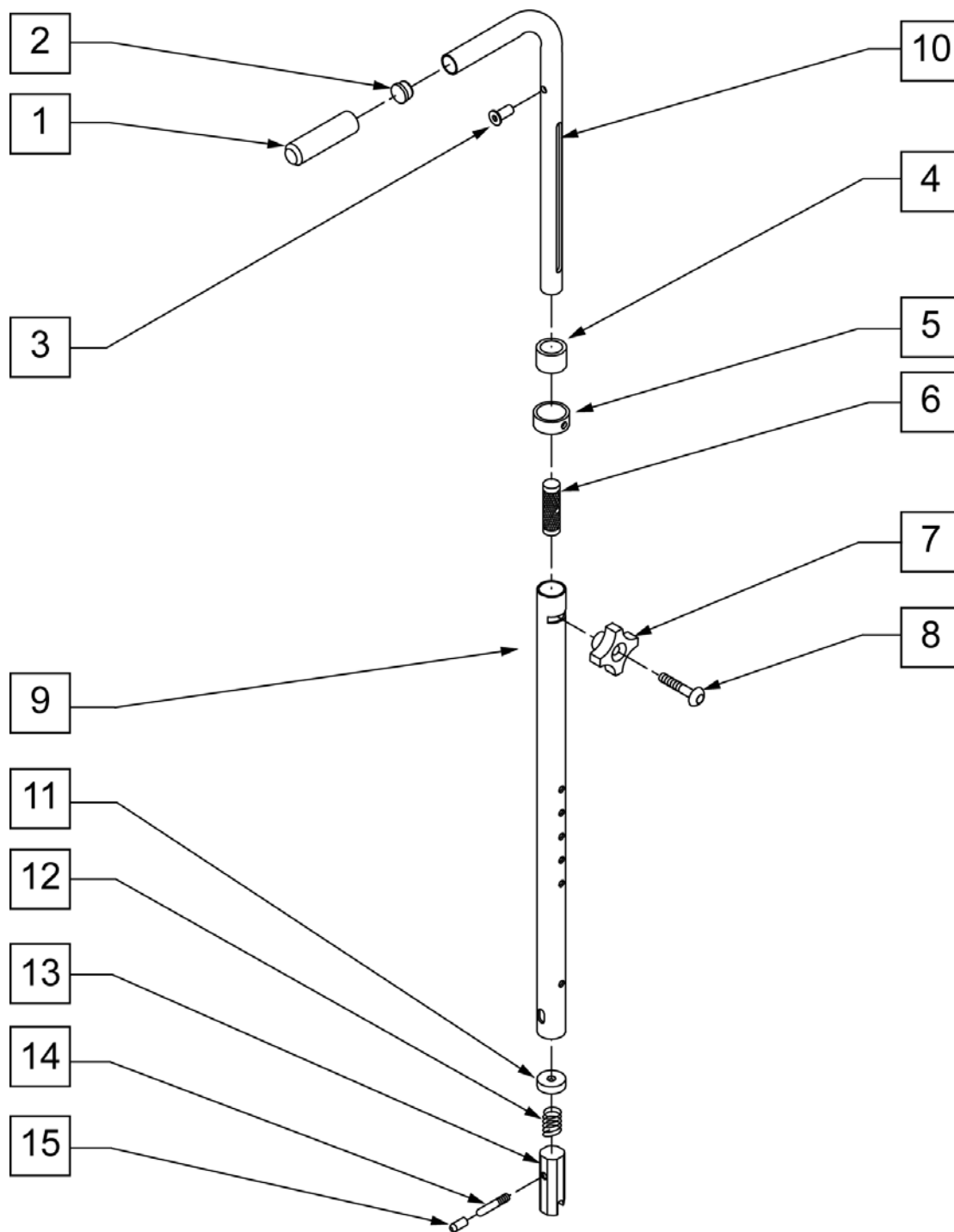

| Pos. | Item Number | Description                   | Remarks |
|------|-------------|-------------------------------|---------|
| 1    | 920026      | QUICKIE GRIP 3/4" W/INSTR SH. |         |
| 2    | 561000-QU   | 3/4" INSERT PLUG .083         |         |
| 3    | 341500      | 10-32 x .010-.080 RIVNUT      |         |
| 4    | 561082      | 1" TUBE END BUSHING THIN      |         |
| 5    | 483096      | S/A ADJ STROLLER RING ZP/QT   |         |

| Pos.  | Item Number | Description                      | Remarks |
|-------|-------------|----------------------------------|---------|
| 6     | 483095      | S/A ADJ STROLLER INSERT ZP/QT    |         |
| 7     | 349018      | KNOB S/A ADJ STROLLER ZP         |         |
| 8     | 226136      | 1/4-20 x 1 1/2 BH C/S BLK        |         |
| 9     | 521791      | S/A ADJ BACK TUBE ONLY ZP/QT     | RIGHT   |
| 9     | 521792      | S/A ADJ BACK TUBE ONLY ZP/QT     | LEFT    |
| 10    | 521793      | ZP S/A ADJ STROLLER TUBE PH R    | RIGHT   |
| 10    | 521794      | ZP S/A ADJ STROLLER TUBE PH L    | LEFT    |
| 11    | 483020      | S/A SPRING RETAINER              |         |
| 12    | 321110      | S/A LATCH SPRING                 |         |
| 13    | 483019      | BACK RELEASE LATCH               |         |
| 14    | 481084      | BACK RELEASE PIN Z2/QT/EX        |         |
| 15    | 560502      | ANTI-TIP LEVER COVER             |         |
| 1-10  | 842082      | S/A ADJ STROLLER HANDLES W/ FOAM | RIGHT   |
| 1-10  | 842083      | S/A ADJ STROLLER HANDLES W/ FOAM | LEFT    |
| 1-10  | 920306      | S/A ADJ STROLLER HANDLES (PR)    | PAIR    |
| 11-15 | 920291      | BACKREST SLIDER LATCH KIT        |         |

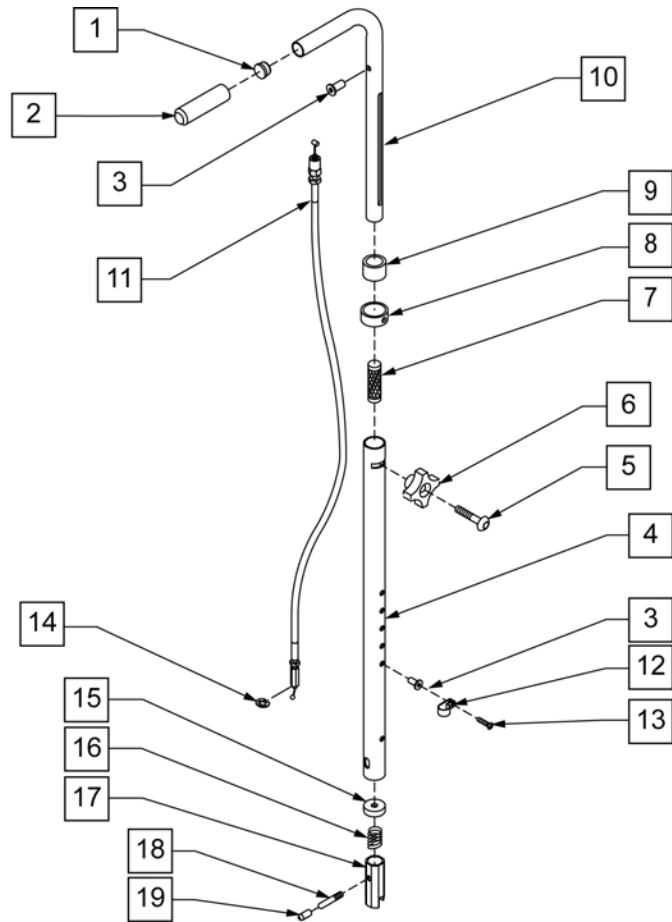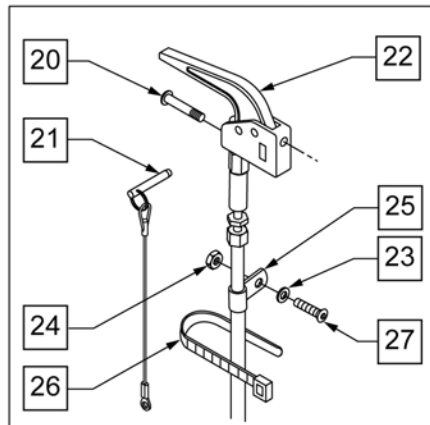


| Pos. | Item Number | Description                       | Remarks |
|------|-------------|-----------------------------------|---------|
| 1    | 561000-QU   | 3/4" INSERT PLUG .083             |         |
| 2    | 920026      | QUICKIE GRIP 3/4" W/INSTR SH.     |         |
| 3    | 341497      | 10-32 x .020-.130 RIVNUT GOLD     |         |
| 4    | 521791      | S/A ADJ BACK TUBE ONLY ZP/QT      | RIGHT   |
| 4    | 521792      | S/A ADJ BACK TUBE ONLY ZP/QT      | LEFT    |
| 5    | 226136      | 1/4-20 x 1 1/2 BH C/S BLK         |         |
| 6    | 349018      | KNOB S/A ADJ STROLLER ZP          |         |
| 7    | 483095      | S/A ADJ STROLLER INSERT ZP/QT     |         |
| 8    | 483096      | S/A ADJ STROLLER RING ZP/QT       |         |
| 9    | 561082      | 1" TUBE END BUSHING THIN          |         |
| 11   | 342057      | CYL LOCK CABLE ASSEMBLY (ZP/TS)   |         |
| 11   | 342056      | CYL LOCK CABLE ASSEMBLY T45 (QTS) |         |
| 12   | 349511      | 1/2" ID PLASTIC CLAMP BLK         |         |
| 13   | 281120      | 10-32 x 5/8 PAN HD PHIL           |         |
| 14   | 342000      | 1/4" E-CLIP                       |         |
| 15   | 483020      | S/A SPRING RETAINER               |         |
| 16   | 321110      | S/A LATCH SPRING                  |         |
| 17   | 483019      | BACK RELEASE LATCH                |         |
| 18   | 481084      | BACK RELEASE PIN Z2/QT/EX         |         |
| 19   | 560502      | ANTI-TIP LEVER COVER              |         |
| 20   | 281120      | 10-32 x 5/8 PAN HD PHIL           |         |
| 21   | 349021      | TRIGGER RELEASE PIN TS/QT/P5      |         |
| 22   | 580110      | CYL LOCK LEVER ASSEMBLY QT/QR/P5  |         |



| Pos.       | Item Number | Description                                 | Remarks |
|------------|-------------|---------------------------------------------|---------|
| 23         | 261204      | #10 AN WASHER .063                          |         |
| 24         | 200100-QU   | 10-32 NYLOCK NUT                            |         |
| 25         | 349511      | 1/2" ID PLASTIC CLAMP BLK                   |         |
| 26         | 560410      | ZIP TIE BLACK                               |         |
| 27         | 226032      | 10-32 x 1 1/2 BH C/S BLK                    |         |
| 1-10       | 842082      | S/A ADJ STROLLER HANDLE W/ FOAM             | RIGHT   |
| 1-10       | 842083      | S/A ADJ STROLLER HANDLE W/ FOAM             | LEFT    |
| 1-10       | 920306      | S/A ADJ STROLLER HANDLES ZP/Z2 (PR)         | PAIR    |
| 1-19       | 920304      | S/A ADJ STROLLER RETRO KIT QTS (PR)         | PAIR    |
| 1-14,20-27 | 842084      | TS ADJ PUSH HANDLE TUBE W/ TRIGGER          | RIGHT   |
| 1-14,20-27 | 842085      | TS ADJ PUSH HANDLE TUBE W/ TRIGGER          | LEFT    |
| 1-14,20-27 | 842073      | QTS ADJ PUSH HANDLE TUBE W/ TRIGGER         | RIGHT   |
| 1-14,20-27 | 842074      | QTS ADJ PUSH HANDLE TUBE W/ TRIGGER         | LEFT    |
| 1-14,20-27 | 920297      | S/A ADJ STROLLER 15"-20" QTS (PR)           | PAIR    |
| 1-14,20-27 | 920307      | S/A ADJ STROLLER 10"-14" TS W/ TRIGGER ON R | PAIR    |
| 1-14,20-27 | 920308      | S/A ADJ STROLLER 15"-20" TS W/ TRIGGER      | PAIR    |
| 15-19      | 920291      | BACKREST SLIDER LATCH KIT                   |         |


NOTE: Stroller handle extension works with 1" tubing backrest only. Example: DAB on Q2 or Standard backrest on positioning chairs.

NOTE: Not compatible with Swing-Away Adjustable Stroller Handle.

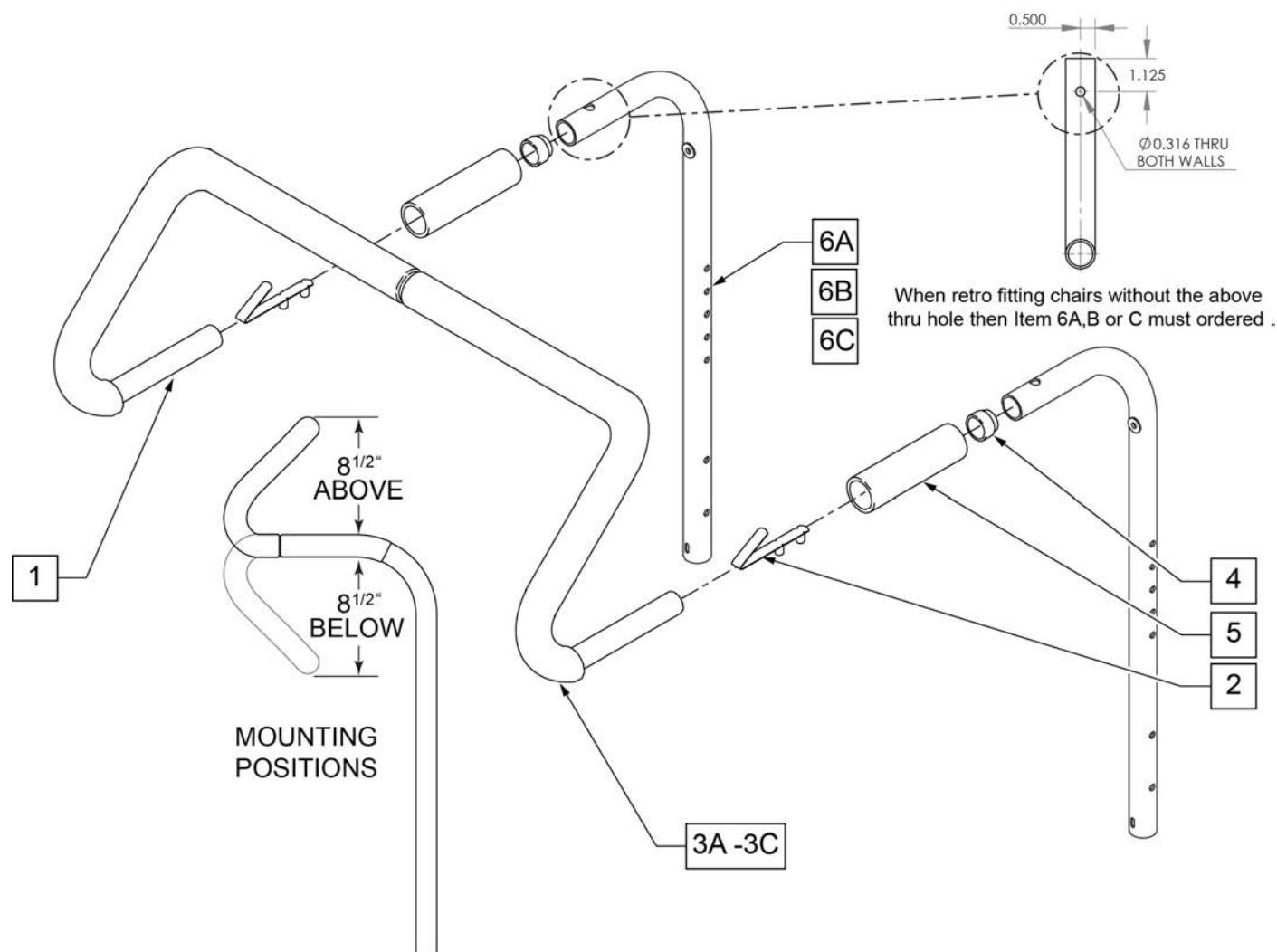


| Pos.     | Item Number | Description                     | Remarks      |
|----------|-------------|---------------------------------|--------------|
| 2        | 301108      | 5/16 SNAP BUTTON DUAL ONE SIDE  |              |
| 3A       | 560615      | STROLLER EXTENSION GRIP 10 -13" | 10 -13" WIDE |
| 3B       | 560616      | STROLLER EXTENSION GRIP 14 -17" | 14 -17" WIDE |
| 3C       | 560617      | STROLLER EXTENSION GRIP 18 -20" | 18 -20" WIDE |
| 4        | 561082      | 1" TUBE END BUSHING THIN        |              |
| 5        | 920023      | QUICKIE GRIP 1 W/ HOLE          |              |
| 1, 2, 3A | 842050      | STROLLER EXTENSION ASSM 10"     |              |
| 1, 2, 3A | 842051      | STROLLER EXTENSION ASSM 11"     |              |
| 1, 2, 3A | 842052      | STROLLER EXTENSION ASSM 12"     |              |
| 1, 2, 3A | 842053      | STROLLER EXTENSION ASSM 13"     |              |
| 1, 2, 3B | 842054      | STROLLER EXTENSION ASSM 14"     |              |

| Pos.     | Item Number | Description                                | Remarks      |
|----------|-------------|--------------------------------------------|--------------|
| 1, 2, 3B | 842055      | STROLLER EXTENSION ASSM 15"                |              |
| 1, 2, 3B | 842056      | STROLLER EXTENSION ASSM 16"                |              |
| 1, 2, 3B | 842057      | STROLLER EXTENSION ASSM 17"                |              |
| 1, 2, 3C | 842058      | STROLLER EXTENSION ASSM 18"                |              |
| 1, 2, 3C | 842059      | STROLLER EXTENSION ASSM 19                 |              |
| 1, 2, 3C | 842063      | STROLLER EXTENSION ASSM 20                 |              |
| 6A       | 842080      | STROLLER HANDLE W/ FOAM STANDARD ZPS/P5/QT | STANDARD/LOW |
| 6B       | 842079      | STROLLER HANDLE W/ FOAM MED ZPS/QT         | MEDIUM       |
| 6C       | 842081      | STROLLER HANDLE W/ FOAM TALL ZPS/QT        | TALL         |

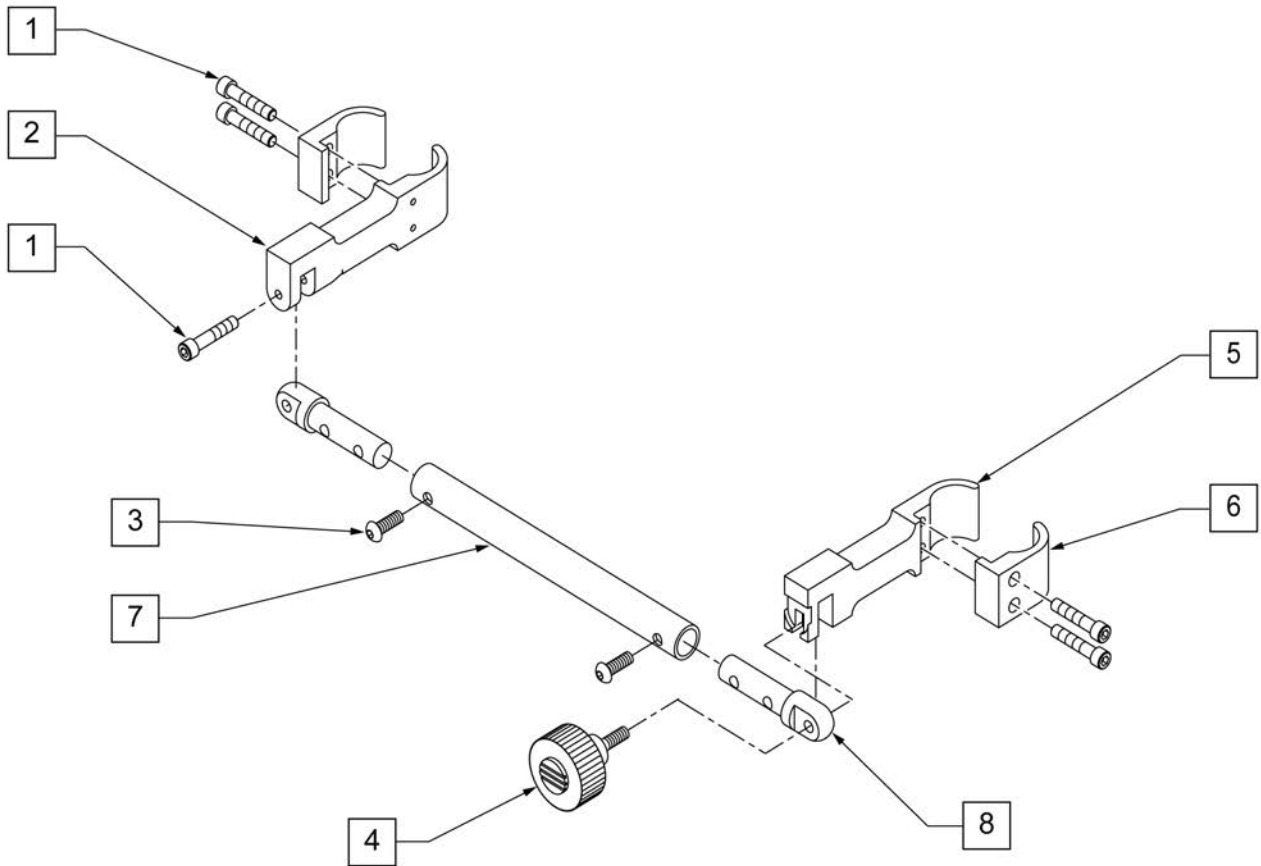


| Pos. | Item Number | Description           |
|------|-------------|-----------------------|
| 1    | 107033      | SHCS 1/4-20 X 1 ROHS  |
| 1-8  | 882056      | RIGIDIZER ZTS 10 -12" |
| 1-8  | 882057      | RIGIDIZER ZTS 12 -14" |
| 1-8  | 882058      | RIGIDIZER ZTS 14 -16" |
| 1-8  | 882059      | RIGIDIZER ZTS 16 -18" |

NOTE: DISCONTINUED ON LX/LXI AS OF 10/24/2008 -PARTS ONLY FOR CHAIRS BUILT PRIOR TO SERIAL NUMBER LX-034330.

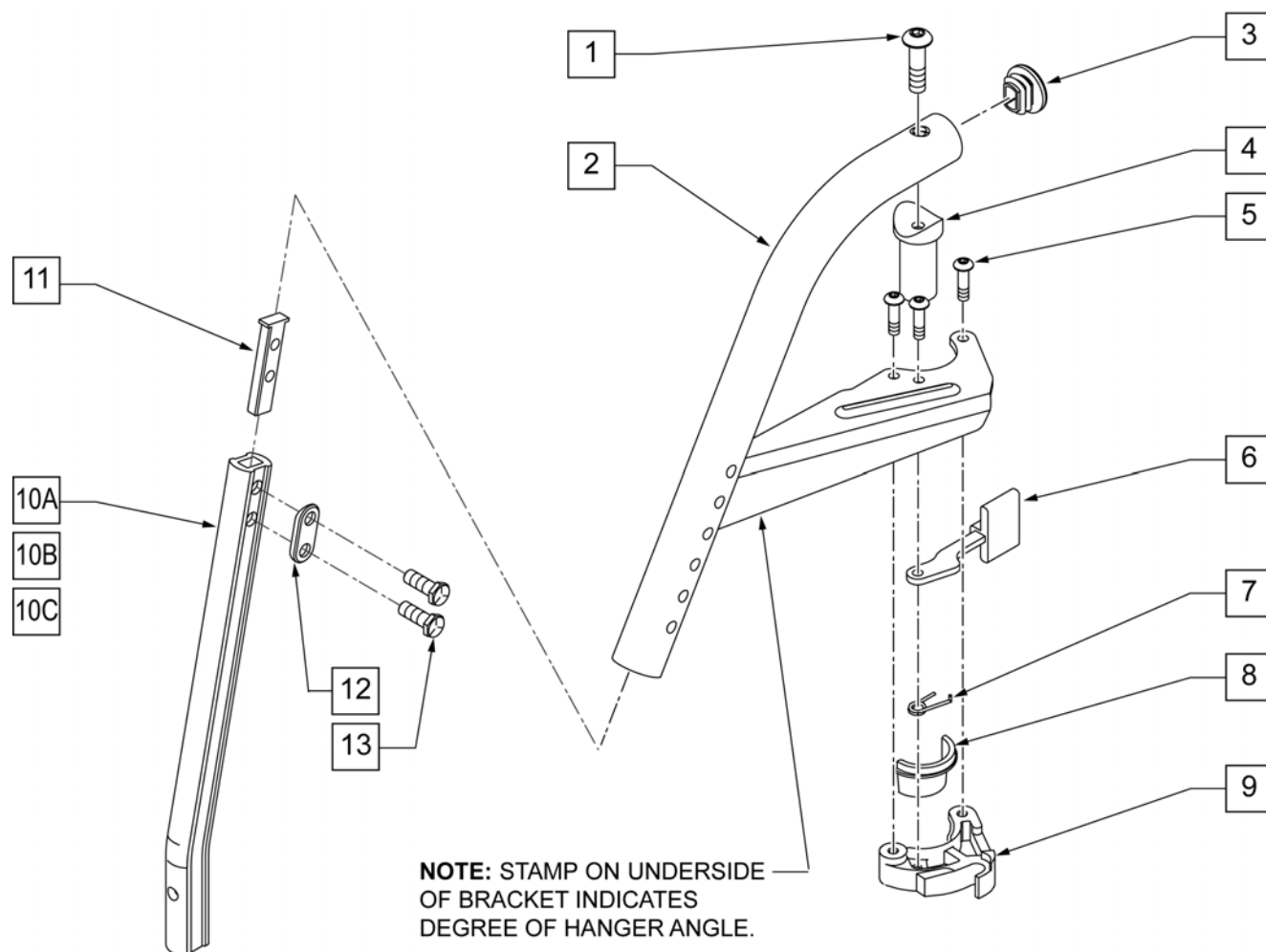


| Pos. | Item Number | Description                          | Remarks |
|------|-------------|--------------------------------------|---------|
| 1    | 226217      | 5/16-18 x 2" BH C/S BLK              |         |
| 3    | 561378      | END PLUG DBL "D" S525                |         |
| 4    | 561197      | PLASTIC PIVOT SADDLE                 |         |
| 5    | 226057      | 10-24 x .725 BH SOC BZ               |         |
| 6    | 561079      | S/A LATCH LEVER                      |         |
| 7    | 321136      | S/A SPRING L RED                     | LEFT    |
| 7    | 321137      | S/A SPRING R CLEAR                   | RIGHT   |
| 8,9  | 881500      | LATCH BLOCK ASSM R                   | RIGHT   |
| 8,9  | 881501      | LATCH BLOCK ASSM L                   | LEFT    |
| 10A  | 550819      | EXT TBE S/A ONLY SHT                 | SHORT   |
| 10B  | 550820      | EXT TBE S/A ONLY MED                 | MEDIUM  |
| 10C  | 550821      | EXT TBE S/A ONLY LONG                | LONG    |
| 11   | 500078      | INSERT D2 EXT TUBE                   |         |
| 12   | 483717      | HANGER SADDLE                        |         |
| 13   | 221102      | 1/4-28 x 3/4 HEX HEAD C/S BLK W/PTCH |         |

| Pos. | Item Number | Description                | Remarks       |
|------|-------------|----------------------------|---------------|
| 1-9  | 641550S     | HANGER 60D R D2 PNT S/A BT | SPECIFY COLOR |
| 1-9  | 641551S     | HANGER 60D L D2 PNT S/A BT | SPECIFY COLOR |
| 5-9  | 881521      | S/A LATCH KIT CASTING      | RIGHT         |
| 5-9  | 881522      | S/A LATCH KIT CASTING      | LEFT          |

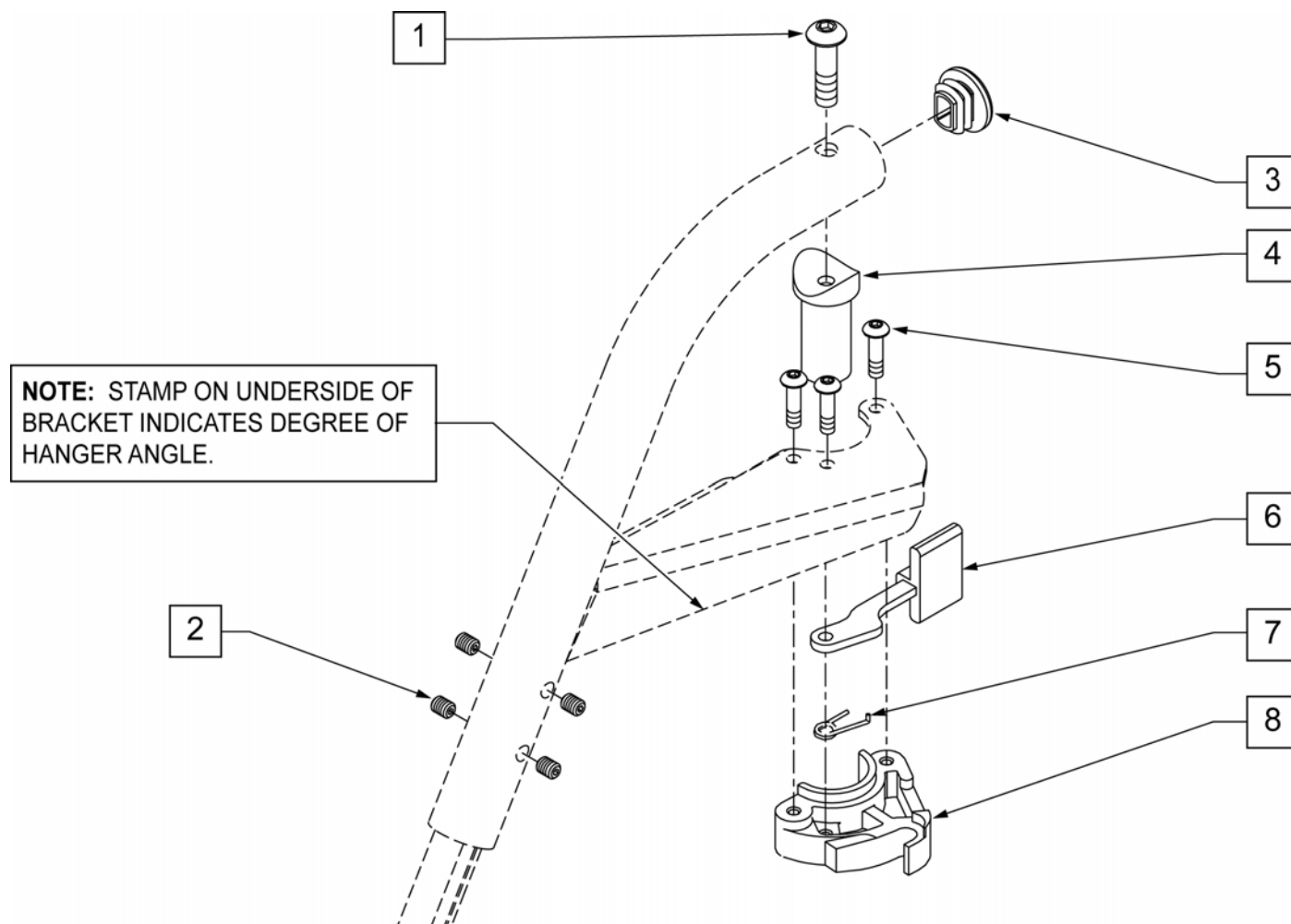

| Pos. | Item Number | Description                     | Remarks |
|------|-------------|---------------------------------|---------|
| 1    | 106983      | 5/16-18 x 2 BH C/S SHT THREAD   |         |
| 2    | 225112      | 1/4-20 x 1/4 SS KNRLD CP W/PTCH |         |
| 3    | 561378      | END PLUG DBL "D" S525           |         |
| 4    | 561197      | PLASTIC PIVOT SADDLE            |         |
| 5    | 226057      | 10-24 x .725 BH SOC BZ          |         |
| 6    | 561079      | S/A LATCH LEVER                 |         |
| 7    | 321136      | S/A SPRING L RED                | LEFT    |
| 7    | 321137      | S/A SPRING R CLEAR              | RIGHT   |
| 8    | 881500      | S/A LATCH BLOCK                 | RIGHT   |
| 8    | 881501      | S/A LATCH BLOCK                 | LEFT    |
| 5-8  | 881521      | S/A LATCH KIT                   | RIGHT   |
| 5-8  | 881522      | S/A LATCH KIT                   | LEFT    |


**NOTE:** THIS HANGER UTILIZES A ROUND EXTENSION TUBE NOT A DOUBLE "D".

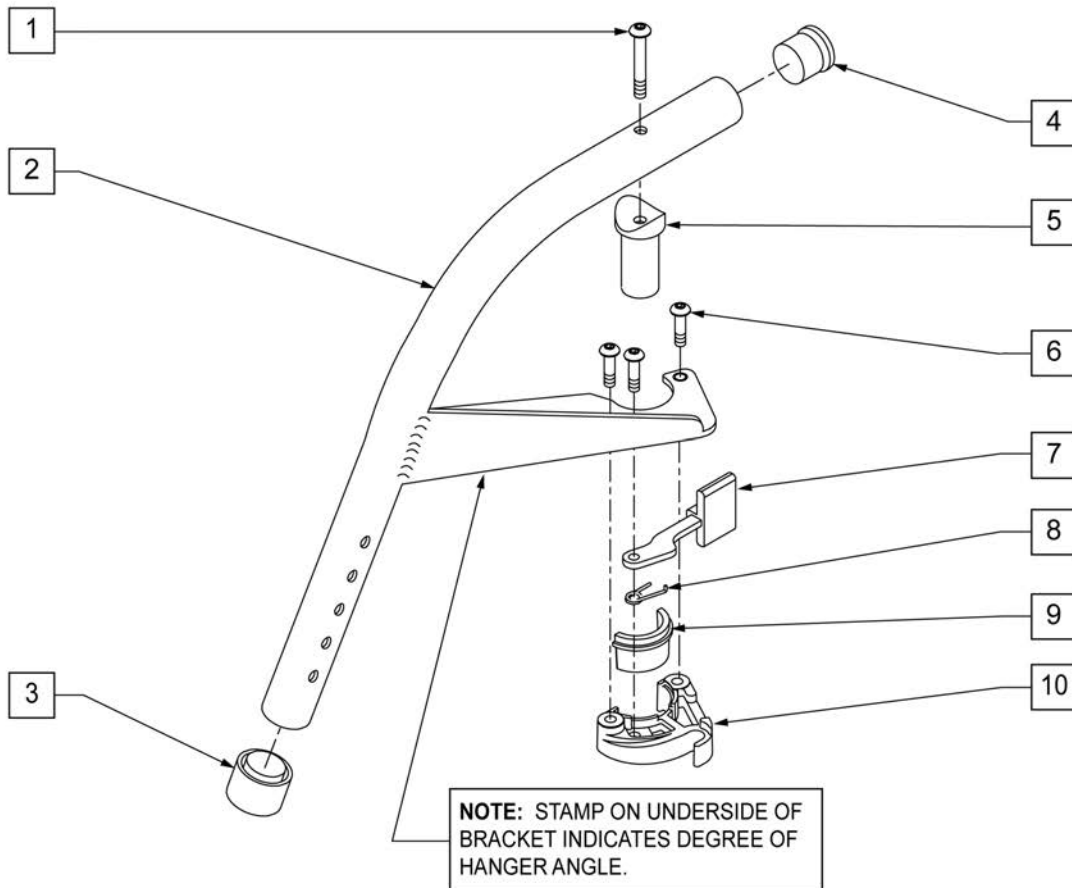

| Pos. | Item Number | Description                            | Remarks |
|------|-------------|----------------------------------------|---------|
| 1    | 106983      | 5/16-18 x 2 BH C/S SHT THREAD          |         |
| 3    | 561082      | 1" TUBE END BUSHING THIN               |         |
| 4    | 561002      | 1" INSERT PLUG .083                    |         |
| 5    | 561197      | PLASTIC PIVOT SADDLE                   |         |
| 1,5  | 880545      | PIVOT SADDLE .125 W/SCREW              |         |
| 6    | 226010      | 10-24 x .725 BH C/S W/PATCH            |         |
| 7    | 561079      | S/A LATCH LEVER                        |         |
| 8    | 321137      | S/A SPRING R CLEAR                     | RIGHT   |
| 8    | 321136      | S/A SPRING L RED                       | LEFT    |
| 9,10 | 881500      | S/A LATCH BLOCK                        | RIGHT   |
| 9,10 | 881501      | S/A LATCH BLOCK                        | LEFT    |
| 6-10 | 881521      | S/A LATCH KIT CASTING RIGHT            | RIGHT   |
| 6-10 | 881522      | S/A LATCH KIT CASTING LEFT             | LEFT    |
| 1-10 | 641150S     | HANGER 60° PAINTED S/A (SPECIFY COLOR) | RIGHT   |
| 1-10 | 641170S     | HANGER 60° PAINTED S/A (SPECIFY COLOR) | LEFT    |




**NOTE:** DISCONTINUED ON LX/LXI AS OF 10/24/2008 -PARTS ONLY FOR CHAIRS BUILT PRIOR TO SERIAL NUMBER LX-034330.

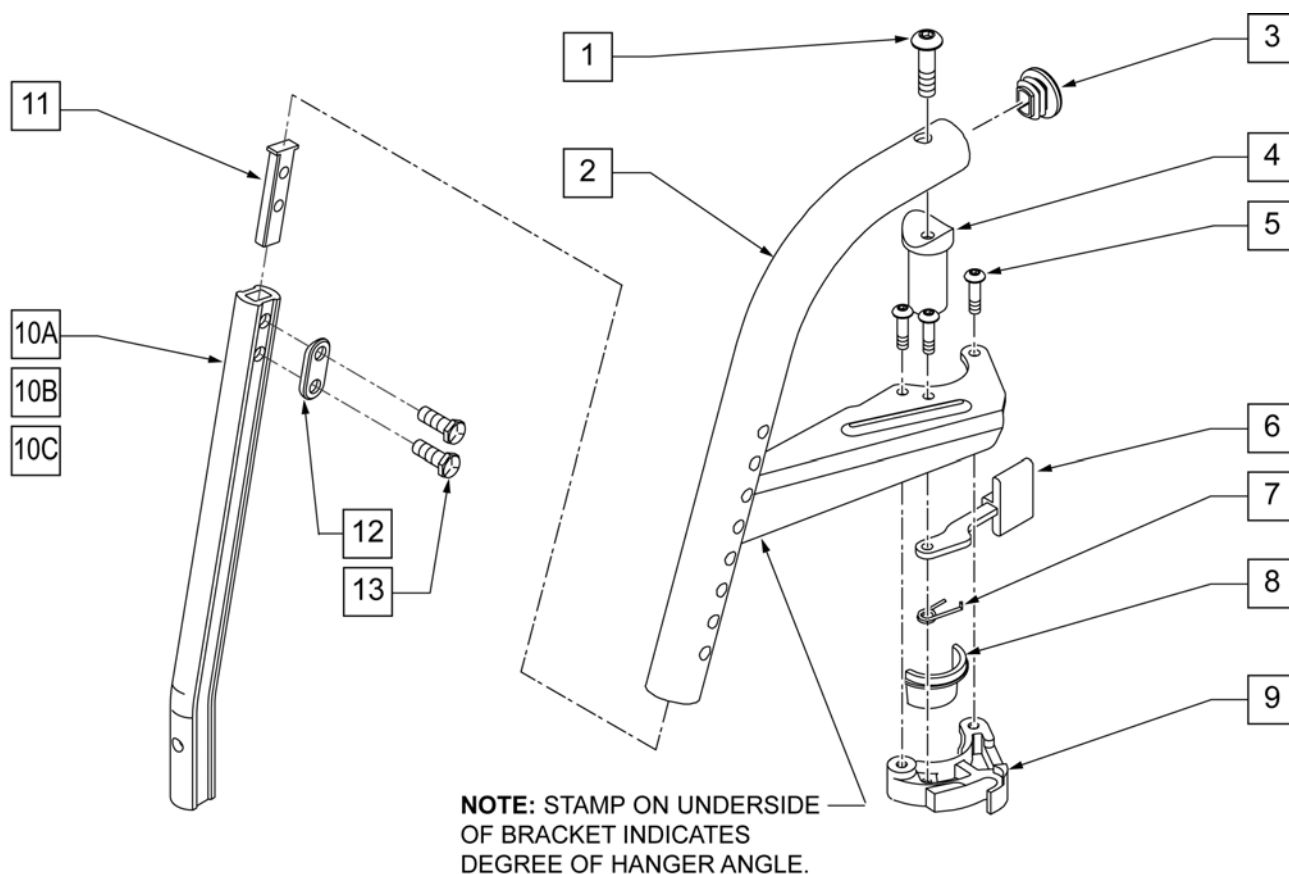

| Pos. | Item Number | Description                          | Remarks       |
|------|-------------|--------------------------------------|---------------|
| 1    | 226217      | 5/16-18 x 2" BH C/S BLK              |               |
| 3    | 561378      | END PLUG DBL-D S525                  |               |
| 4    | 561197      | PLASTIC PIVOT SADDLE                 |               |
| 5    | 226057      | 10-24 x .725 BH SOC BZ               |               |
| 6    | 561079      | S/A LATCH LEVER                      |               |
| 7    | 321136      | S/A SPRING L RED                     | LEFT          |
| 7    | 321137      | S/A SPRING R CLEAR                   | RIGHT         |
| 8,9  | 881500      | LATCH BLOCK ASSM R                   | RIGHT         |
| 8,9  | 881501      | LATCH BLOCK ASSM L                   | LEFT          |
| 10A  | 550819      | EXT TBE S/A ONLY SHT                 |               |
| 10B  | 550820      | EXT TBE S/A ONLY MED                 |               |
| 10C  | 550821      | EXT TBE S/A ONLY LONG                |               |
| 11   | 500078      | INSERT D2 EXT TUBE                   |               |
| 12   | 483717      | HANGER SADDLE                        |               |
| 13   | 221102      | 1/4-28 x 3/4 HEX HEAD C/S BLK W/PTCH |               |
| 1-9  | 641552S     | HANGER 70D R D2 PNT S/A BT           | SPECIFY COLOR |
| 1-9  | 641553S     | HANGER 70D L D2 PNT S/A BT           | SPECIFY COLOR |
| 5-9  | 881521      | S/A LATCH KIT CASTING                | RIGHT         |
| 5-9  | 881522      | S/A LATCH KIT CASTING                | LEFT          |


**NOTE:** IF REPLACING AN EXTENSION TUBE ON AN MANUAL ALR PLEASE REFER TO THE ROUND EXT TUBE PARTS MANUAL PAGE.

**NOTE:** WHEN ORDERING REPLACEMENT EXTENSION TUBES PLEASE VERIFY IF YOU HAVE THE OLD STYLE DOUBLE D EXTENSION TUBE OR THE NEW STYLE DOUBLE D EXTENSION TUBE. THIS CAN BE DONE BY REVIEWING THE HANGER THE EXTENSION TUBE IS USED ON. IF THE HANGER UTILIZES A SINGLE SET SCREW AT THE BOTTOM OF THE HANGER TUBE TO ATTACH THE EXTENSION TUBE TO THE HANGER, THEN THIS UTILIZES THE NEW STYLE DOUBLE D EXT TUBE ALL OTHER HANGERS REQUIRING A DOUBLE D EXTENSION TUBE UTILIZE THE OLD STYLE.

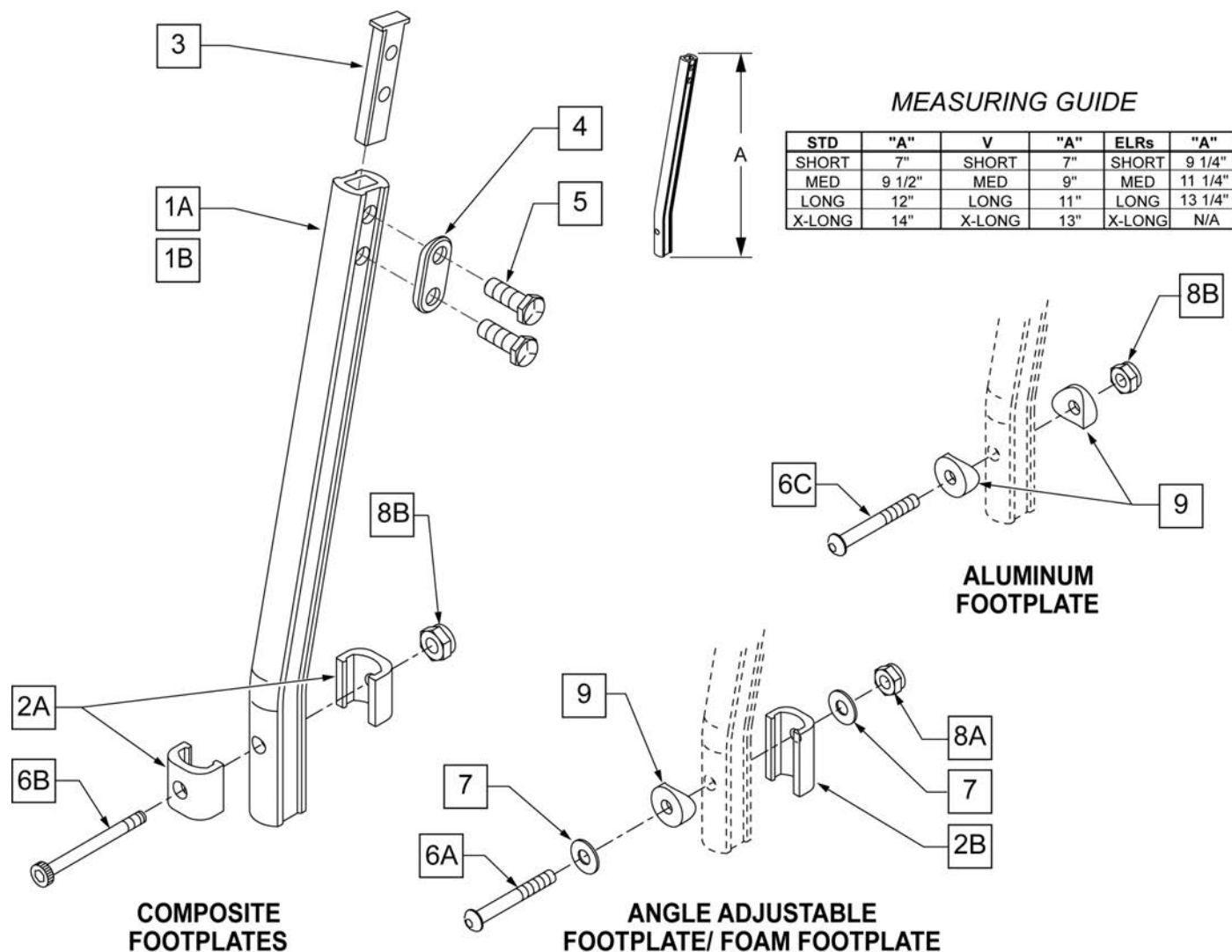


| Pos. | Item Number | Description                 | Remarks                           |
|------|-------------|-----------------------------|-----------------------------------|
| 1A   | 550819      | EXT TUBE S/A ONLY SHT       | Short Ext-Swing Away Hanger       |
| 1A   | 550820      | EXT TUBE S/A ONLY MED       | Med Ext-Swing Away Hanger         |
| 1A   | 550821      | EXT TUBE S/A ONLY LONG      | Long Ext-Swing Away Hanger        |
| 1A   | 550822      | EXT TUBE S/A ONLY X-LONG    | X-Long Ext-Swing Away Hanger      |
| 1B   | 550834      | EXT TUBE V IN ONLY D2 SHT R | Right Short Ext-Swing Away Hanger |
| 1B   | 550835      | EXT TUBE V IN ONLY D2 SHT L | Left Short Ext-Swing Away Hanger  |

| Pos.      | Item Number | Description                          | Remarks                                 |
|-----------|-------------|--------------------------------------|-----------------------------------------|
| 1B        | 550836      | EXT TUBE V IN ONLY D2 MED R          | Right Med Ext-Swing Away Hanger         |
| 1B        | 550837      | EXT TUBE V IN ONLY D2 MED L          | Left Med Ext-Swing Away Hanger          |
| 1B        | 550838      | EXT TUBE V IN ONLY D2 LONG R         | Right Long Ext-Swing Away Hanger        |
| 1B        | 550839      | EXT TUBE V IN ONLY D2 LONG L         | Left Long Ext-Swing Away Hanger         |
| 1B        | 550840      | EXT TUBE V IN ONLY D2 X-LONG R       | Right X-Long Ext-Swing Away Hanger      |
| 1B        | 550841      | EXT TUBE V IN ONLY D2 X-LONG L       | Left X-Long Ext-Swing Away Hanger       |
| 2A        | 483718      | HANGER EXTENSION SADDLE              | USE W/ COMPOSITE FTPLT                  |
| 2B        | 483720      | HANGER EXTENSION SADDLE LONG         | USE W/ ANGLE ADJ FTPLT                  |
| 1C        | 550764      | S/A HANGER EXT D2 SHORT BLACK        | Short Ext ELR Hangers                   |
| 1C        | 550765      | S/A HANGER EXT D2 MED BLACK          | Medium Ext ELR Hangers                  |
| 1C        | 550766      | S/A HANGER EXT D2 LONG BLACK         | Long Ext ELR Hangers                    |
| 3         | 500078      | INSERT D2 EXT TUBE                   |                                         |
| 4         | 483717      | HANGER SADDLE                        |                                         |
| 5         | 221102      | 1/4-28 x 3/4 HEX HEAD C/S BLK W/PTCH |                                         |
| 6A        | 106962      | 1/4-28 x 1 3/4 BH C/S BLK ZN         | USE W/ ANGLE ADJ FTPLT                  |
| 6B        | 226165      | 1/4 -28 x 2 1/2 SOC C/S              | USE W/ COMPOSITE FTPLT                  |
| 6C        | 226147      | 1/4 -28 X 1.85 BH C/S                |                                         |
| 7         | 106945      | 1/4" SAE WASHER                      |                                         |
| 8A        | 106941      | 1/4 -28 AUTOMATION NUT               | USE W/ ANGLE ADJ FTPLT                  |
| 8B        | 106935      | 1/4 -28 NYLOCK NUT THIN              | USE W/ COMPOSITE FTPLT                  |
| 9         | 561055      | 3/4 SADDLE 1" OD                     | USE W/ ANGLE ADJ FTPLT                  |
| 1A,2A-5   | 921759      | EXT TUBE S/A SHT ASSM PCKT           | USE W/ COMPOSITE FTPLT                  |
| 1A,2A-5   | 921760      | EXT TUBE S/A MED ASSM PCKT           | USE W/ COMPOSITE FTPLT                  |
| 1A,2A-5   | 921761      | EXT TUBE S/A LONG ASSM PCKT          | USE W/ COMPOSITE FTPLT                  |
| 1A,2A-5   | 921762      | EXT TUBE S/A X-LONG ASSM PCKT        | USE W/ COMPOSITE FTPLT                  |
| 1B,2A-5   | 921751      | EXT TUBE "V" D2 SHT R ASSM PCKT      | RIGHT - USE W/ COMPOSITE FTPLT V HANGER |
| 1B,2A-5   | 921752      | EXT TUBE "V" D2 SHT L ASSM PCKT      | LEFT - USE W/ COMPOSITE FTPLT V HANGER  |
| 1B,2A-5   | 921753      | EXT TUBE "V" D2 MED R ASSM PCKT      | RIGHT - USE W/ COMPOSITE FTPLT V HANGER |
| 1B,2A-5   | 921754      | EXT TUBE "V" D2 MED L ASSM PCKT      | LEFT - USE W/ COMPOSITE FTPLT V HANGER  |
| 1B,2A-5   | 921755      | EXT TUBE "V" D2 LONG R ASSM PCKT     | RIGHT - USE W/ COMPOSITE FTPLT V HANGER |
| 1B,2A-5   | 921756      | EXT TUBE "V" D2 LONG L ASSM PCKT     | LEFT - USE W/ COMPOSITE FTPLT V HANGER  |
| 1B,2A-5   | 921757      | EXT TUBE "V" D2 X-LONG R ASSM PCKT   | RIGHT - USE W/ COMPOSITE FTPLT V HANGER |
| 1B,2A-5   | 921758      | EXT TUBE "V" D2 X-LONG L ASSM PCKT   | LEFT - USE W/ COMPOSITE FTPLT V HANGER  |
| 1A,2A-7,9 | 921772      | EXT TUBE S/A SHT ANG ADJ PCKT        | USE W/ ANGLE ADJ FTPLT                  |
| 1A,2A-7,9 | 921773      | EXT TUBE S/A MED ANG ADJ PCKT        | USE W/ ANGLE ADJ FTPLT                  |
| 1A,2A-7,9 | 921774      | EXT TUBE S/A LONG ANG ADJ PCKT       | USE W/ ANGLE ADJ FTPLT                  |
| 1A,2A-7,9 | 921775      | EXT TUBE S/A X-LONG ANG ADJ PCKT     | USE W/ ANGLE ADJ FTPLT                  |
| 1B,2A-7,9 | 921764      | EXT TUBE "V" D2 SHT R ANG ADJ PCKT   | RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER |
| 1B,2A-7,9 | 921765      | EXT TUBE "V" D2 SHT L ANG ADJ PCKT   | LEFT - USE W/ ANGLE ADJ FTPLT V HANGER  |
| 1B,2A-7,9 | 921766      | EXT TUBE "V" D2 MED R ANG ADJ PCKT   | RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER |
| 1B,2A-7,9 | 921767      | EXT TUBE "V" D2 MED L ANG ADJ PCKT   | LEFT - USE W/ ANGLE ADJ FTPLT V HANGER  |
| 1B,2A-7,9 | 921768      | EXT TUBE "V" D2 LONG R ANG ADJ PCKT  | RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER |
| 1B,2A-7,9 | 921769      | EXT TUBE "V" D2 LONG L ANG ADJ PCKT  | LEFT - USE W/ ANGLE ADJ FTPLT V HANGER  |
| 1B,2A-7,9 | 921770      | EXT TUBE "V" D2 X-LONG R ANG AD PCKT | RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER |
| 1B,2A-7,9 | 921771      | EXT TUBE "V" D2 X-LONG L ANG AD PCKT | LEFT - USE W/ ANGLE ADJ FTPLT V HANGER  |

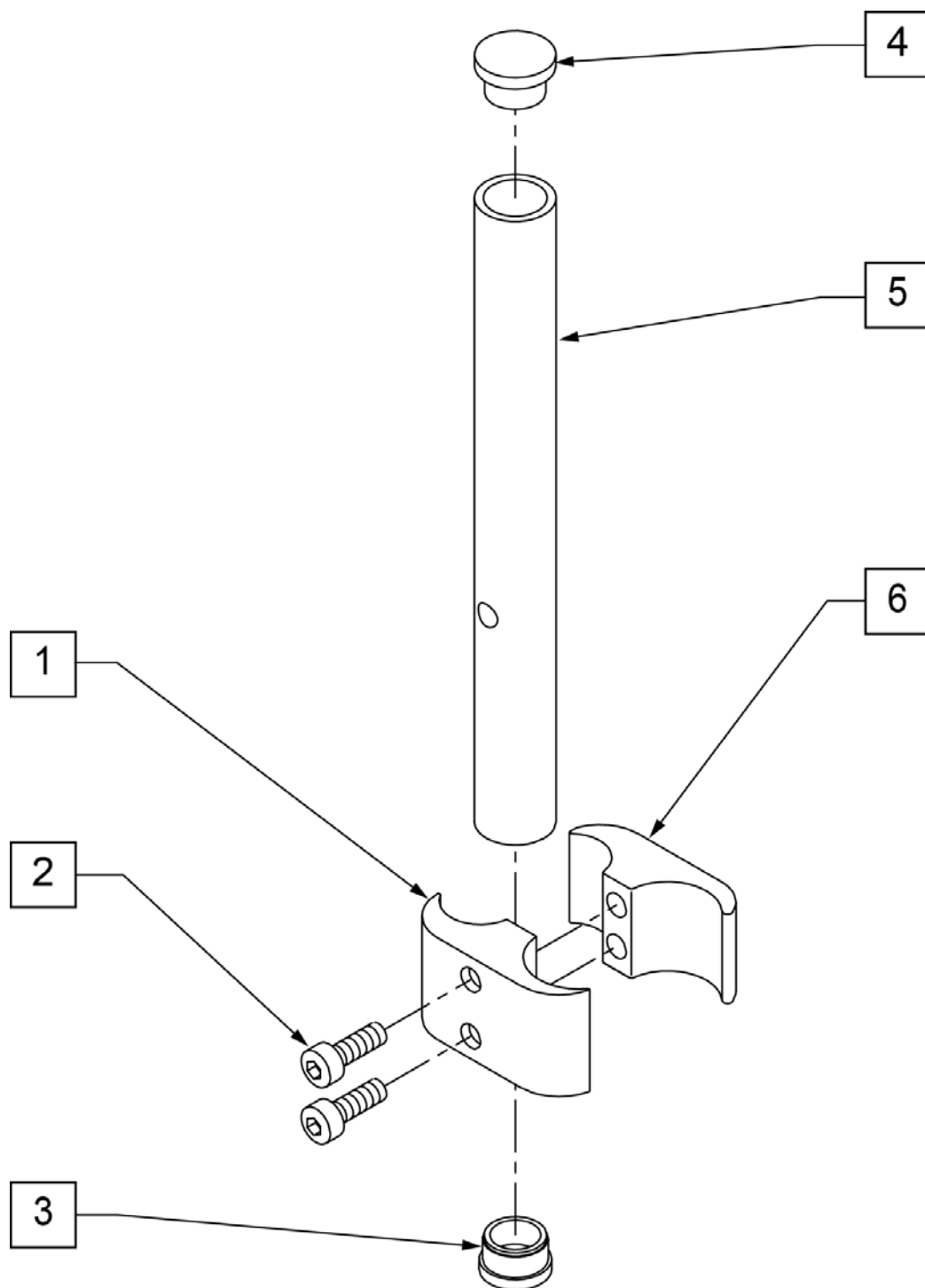

| Pos.  | Item Number | Description                        | Remarks |
|-------|-------------|------------------------------------|---------|
| 1     | 550657      | CLAMP 1 1/4" RECESS FULL BLK       |         |
| 2     | 107032      | 1/4-28 x 1 SOC C/S BLK             |         |
| 3     | 561000-QU   | 3/4" INSERT PLUG .083              |         |
| 4     | 482016      | H/M FL FOOT TUBE PLUG              |         |
| 5     | 521100      | H/M FL FOOT EXTENSION TUBE         |         |
| 6     | 550658      | CLAMP 1 1/4" THREADED BLK          |         |
| 1,2,6 | 880538      | HIGH MOUNT FR CLAMP ASSEMBLY CLEAR |         |
| 1-6   | 921550S     | HIGH MOUNT FL FOOT CLAMPS & TUBES  | PAIR    |

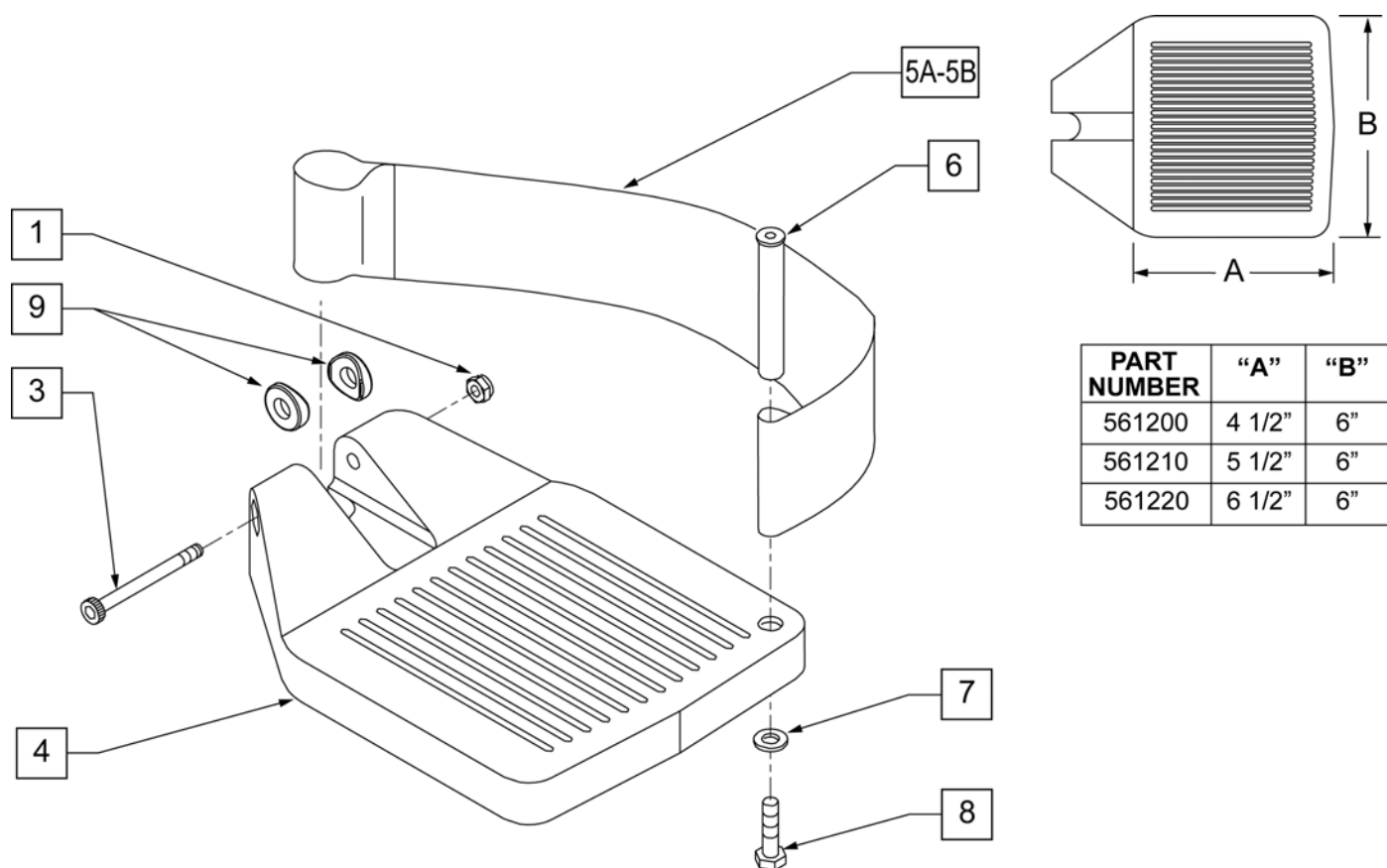

| Pos.  | Item Number | Description                                   | Remarks                                 |
|-------|-------------|-----------------------------------------------|-----------------------------------------|
| 1     | 106935      | 1/4 - 28 NYLOCK NUT                           |                                         |
| 3     | 226165      | 1/4 - 28 x 2 1/2 SOC C/S                      |                                         |
| 4     | 561200      | COMPOSITE FOOTPLATE 14 -15 (16 -18 V )        | 14 -15 OR 16 -18V                       |
| 4     | 561210      | COMPOSITE FOOTPLATE 16 -17 (19 -20 V )        | 16 -17 OR 19 -20V                       |
| 4     | 561220      | COMPOSITE FOOTPLATE 18 -20                    | 18 -20                                  |
| 5A    | 741151      | HEEL LOOP 14-15 COMPOSITE (14 -15 or 16 -18V) | 14 -15 OR 16 -18V                       |
| 5A    | 741156      | HEEL LOOP 16 -17 or 19 -20V                   | 16 -17 OR 19 -20V                       |
| 5A    | 741153      | HEEL LOOP 18-20 COMPOSITE/14-15 ANG ADJ       | 18 -20                                  |
| 5A    | 902270      | HEEL LOOP COMPOSITE FT 14-15                  | PAIR 14 -15 OR 16 -18V                  |
| 5A    | 902271      | HEEL LOOP COMPOSITE FT 16-17                  | PAIR 16 -17 OR 19 -20V                  |
| 5A    | 902272      | HEEL LOOP COMPOSITE FT 18-20                  | PAIR 18 -20                             |
| 5B    | 741154      | HEEL LOOP 14-15 COMPOSITE HEMI                | 14 -15 HEMI                             |
| 5B    | 741155      | HEEL LOOP 16-17 COMPOSITE HEMI                | 16 -17 HEMI                             |
| 5B    | 741153      | HEEL LOOP 18-20 COMPOSITE HEMI                | 18 -10 HEMI                             |
| 5B    | 902260      | HEEL LOOPS HEMI COMPOSITE 14-15               | PAIR 14 -15 HEMI                        |
| 5B    | 902261      | HEEL LOOPS HEMI COMPOSITE 16-17               | PAIR 16 -17 HEMI                        |
| 5B    | 902262      | HEEL LOOPS HEMI COMPOSITE 18-20               | PAIR 18 -20 HEMI                        |
| 6     | 561040      | HEEL LOOP PEG                                 |                                         |
| 7     | 106944      | 1/4 AN WASHER FLAT THIN                       |                                         |
| 8     | 106958      | HHCS 1/4-20 X 1 ROHS                          |                                         |
| 9     | 561003      | 3/4" SADDLE THIN                              | USED W/ ROUND EXT TUBES ONLY            |
| 1,3,4 | 921180      | COMPOSITE FOOTPLATE ASSM 14-15                | RIGHT (14 -15 OR 16 -18V) W/O HEEL LOOP |
| 1,3,4 | 921181      | COMPOSITE FOOTPLATE ASSM 14-15                | LEFT (14 -15 OR 16 -18V) W/O HEEL LOOP  |
| 1,3,4 | 921182      | COMPOSITE FOOTPLATE ASSM 16-17                | RIGHT (16 -17 OR 19 -20V) W/O HEEL LOOP |
| 1,3,4 | 921183      | COMPOSITE FOOTPLATE ASSM 16-17                | LEFT (16 -17 OR 19 -20V) W/O HEEL LOOP  |

| Pos.     | Item Number | Description                                | Remarks                                |
|----------|-------------|--------------------------------------------|----------------------------------------|
| 1,3,4    | 921184      | COMPOSITE FOOTPLATE ASSM 18-20             | RIGHT (18 -20) W/O HEEL LOOP           |
| 1,3,4    | 921185      | COMPOSITE FOOTPLATE ASSM 18-20             | LEFT (18 -20) W/O HEEL LOOP            |
| 1,3,4    | 921151      | COMPOSITE FOOTPLATE ASSM 14-15/16-18V      | PAIR (14 -15 OR 16 -18V) W/O HEEL LOOP |
| 1,3,4    | 921152      | COMPOSITE FOOTPLATE ASSM 16-17/19-20V      | PAIR (16 -17 OR 19 -20V) W/O HEEL LOOP |
| 1,3,4    | 921153      | COMPOSITE FOOTPLATE ASSM 18-20             | PAIR (18 -20) W/O HEEL LOOP            |
| 1,3,4A-8 | 921190      | COMPOSITE FOOTPLATE ASSM HL LP 14-15       | RIGHT (14 -15 OR 16 -18V) W/ HEEL LOOP |
| 1,3,4A-8 | 921191      | COMPOSITE FOOTPLATE ASSM HL LP 14-15       | LEFT (14 -15 OR 16 -18V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921192      | COMPOSITE FOOTPLATE ASSM HL LP 16-17       | RIGHT (16 -17 OR 19 -20V) W/ HEEL LOOP |
| 1,3,4A-8 | 921193      | COMPOSITE FOOTPLATE ASSM HL LP 16-17       | LEFT (16 -17 OR 19 -20V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921194      | COMPOSITE FOOTPLATE ASSM HL LP 18-20       | RIGHT (18 -20) W/ HEEL LOOP            |
| 1,3,4A-8 | 921195      | COMPOSITE FOOTPLATE ASSM HL LP 18-20       | LEFT (18 -20) W/ HEEL LOOP             |
| 1,3,4A-8 | 921161      | COMPOSITE FOOTPLATE ASSM HL 14-15/16-18V   | PAIR (14 -15 OR 16 -18V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921162      | COMPOSITE FOOTPLATE ASSM HL 16-17/19-20V   | PAIR (16 -17 OR 19 -20V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921163      | COMPOSITE FOOTPLATE ASSM HL 18-20          | PAIR (18 -20) W/ HEEL LOOP             |
| 1,3,4B-8 | 921140      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15  | RIGHT (14 -15) W/ HEEL LOOP            |
| 1,3,4B-8 | 921141      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15  | LEFT (14 -15) W/ HEEL LOOP             |
| 1,3,4B-8 | 921142      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17  | RIGHT (16 -17) W/ HEEL LOOP            |
| 1,3,4B-8 | 921143      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17  | LEFT (16 -17) W/ HEEL LOOP             |
| 1,3,4B-8 | 921144      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20  | RIGHT (18 -20) W/ HEEL LOOP            |
| 1,3,4B-8 | 921145      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20  | LEFT (18 -20) W/ HEEL LOOP             |
| 1,3,4B-8 | 921167      | COMPOSITE FOOTPLATE ASSM HEMI HL LPS 14-15 | PAIR (14 -15) W/ HEEL LOOP             |
| 1,3,4B-8 | 921168      | COMPOSITE FOOTPLATE ASSM HEMI HL LPS 16-17 | PAIR (16 -17) W/ HEEL LOOP             |
| 1,3,4B-8 | 921169      | COMPOSITE FOOTPLATE ASSM HEMI HL LPS 18-20 | PAIR (18 -20) W/ HEEL LOOP             |


**NOTE:** FOR HIGH-MOUNT ADULT ANGLE-ADJUSTABLE FOOTPLATE, SEE "HIGH-MOUNT EXTENSION TUBE" PAGE.

**NOTE:** ORDER ITEM 4B FOR CHAIRS PRIOR TO 8/13/01.

**NOTE:** ONLY 1 WASHER PRIOR TO 8/13/01.

**NOTE:** ORDER PART 226167 PRIOR TO 8/13/01.

**NOTE:** See attached REFERENCE GUIDE.

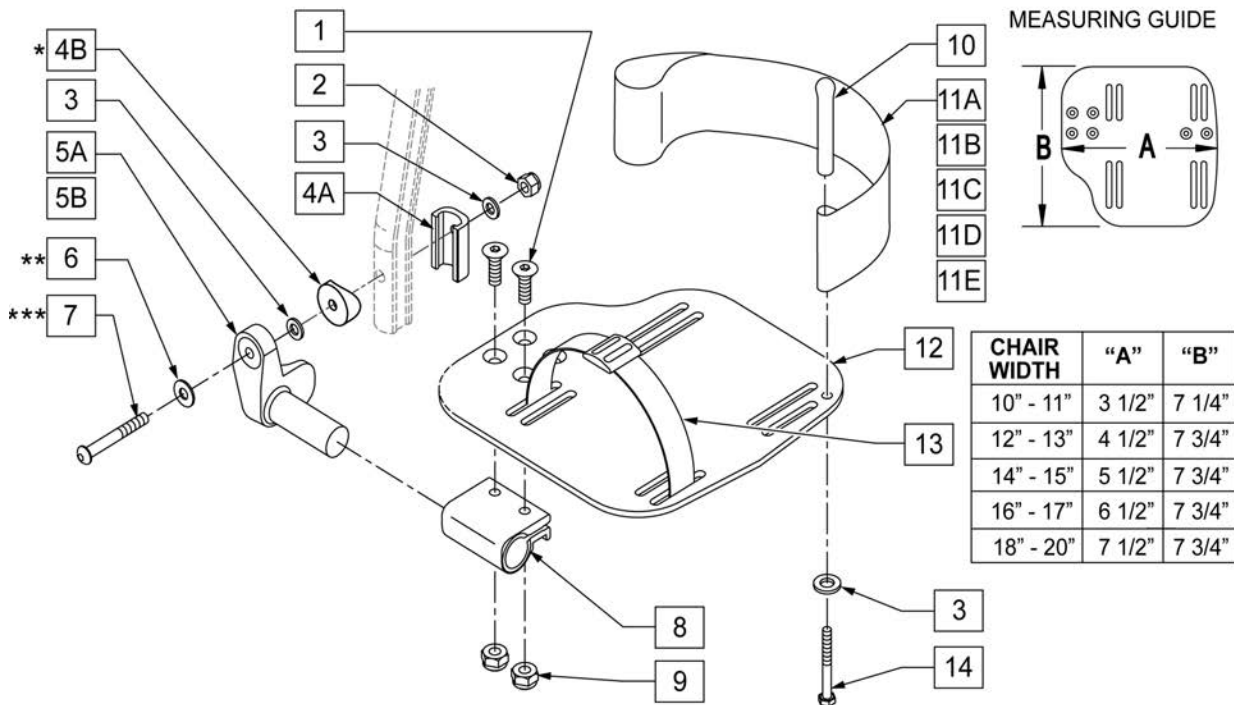


| Pos. | Item Number | Description           | Remarks |
|------|-------------|-----------------------|---------|
| 1    | 106978      | 1/4-28 x 7/8 FH C/S   |         |
| 2    | 106941      | 1/4-28 AUTOMATION NUT |         |



| Pos.                    | Item Number | Description                         | Remarks                                    |
|-------------------------|-------------|-------------------------------------|--------------------------------------------|
| 3                       | 106943      | WASHER FLAT AN 1/4 ROHS             |                                            |
| 4A                      | 483720      | HANGER EXTENSION SADDLE LONG        | USE W/ DOUBLE D EXT TUBE                   |
| 4B                      | 561055      | 3/4" SADDLE 1" OD                   | USE W/ ROUND EXT TUBE                      |
| 5                       | 541002      | FTRST MOUNTING BRKT-3/8 RAD R       | RIGHT                                      |
| 5                       | 541003      | FTRST MOUNTING BRKT-3/8 RAD L       | LEFT                                       |
| 6                       | 106945      | WASHER FLAT SAE 1/4 ROHS            |                                            |
| 7A                      | 106962      | BHCS 1/4-28 X 1-3/4 ROHS            | USE W/ DOUBLE D EXT TUBE                   |
| 7B                      | 226167      | 1/4-28 x 1.625 BH C/S BLK           | USE W/ ROUND EXT TUBE                      |
| 8                       | 550350      | 3/4" CLMP ADJ FT PLT                |                                            |
| 9                       | 106935      | NUT NYLOCK THIN 1/4-28 ROHS         |                                            |
| 10                      | 561040      | HEEL LOOP PEG                       |                                            |
| 11A                     | 741154      | HEEL LOOP 14-15 CMPST HEMI          | USED W/ 10-11" ADULT ANG ADJ FTPLTS        |
| 11B                     | 741155      | HEEL LOOP 16-17 CMPST HEMI          | USED W/ 12 -13" ADULT ANG ADJ FTPLTS       |
| 11C                     | 741153      | H/L 18-20 CMPST/14-15 ANG ADJ       | USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS |
| 11D                     | 741253      | HEEL LOOP ADLT ADJ F/PLT 18-20      | USED W/ 18 -20" ADULT ANG ADJ FTPLTS       |
| 12                      | 500656      | ADLT ANGL ADJ F/PLT 10/11" R        | RIGHT                                      |
| 12                      | 500657      | ADLT ANGL ADJ F/PLT 10/11" L        | LEFT                                       |
| 12                      | 500658      | ADLT ANGL ADJ F/PLT 12/13" R        | RIGHT                                      |
| 12                      | 500659      | ADLT ANGL ADJ F/PLT 12/13" L        | LEFT                                       |
| 12                      | 500660      | ADLT ANGL ADJ F/PLT 14/15           | RIGHT                                      |
| 12                      | 500661      | ADLT ANGL ADJ F/PLT 14/15           | LEFT                                       |
| 12                      | 500662      | ADLT ANGL ADJ F/PLT 16/17           | RIGHT                                      |
| 12                      | 500663      | ADLT ANGL ADJ F/PLT 16/17           | LEFT                                       |
| 12                      | 500664      | ADLT ANGL ADJ F/PLT 18-20           | RIGHT                                      |
| 12                      | 500665      | ADLT ANGL ADJ F/PLT 18-20           | LEFT                                       |
| 13                      | 740260      | TOE/ANKLE STRAP ADJ ADLT F/PLT      |                                            |
| 14                      | 106958      | 1/4-20 x 1 HEX C/S                  |                                            |
| 1-3,5,6,8,9,12          | 820526      | AD ANG ADJ F/PLT R 12"/13" OMIT HL  | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820536      | AD ANG ADJ F/PLT L 12"/13" OMT HL   | LEFT                                       |
| 1-3,5,6,8,9,12          | 820460      | FP ADLT ADJ 14/15 OMIT HL           | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820462      | FP ADLT ADJ 14/15 OMIT HL           | LEFT                                       |
| 1-3,5,6,8,9,12          | 820464      | FP ADLT ADJ 16/17 OMIT HL           | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820466      | FP ADLT ADJ 16/17 OMIT HL           | LEFT                                       |
| 1-3,5,6,8,9,12          | 820468      | FP ADLT ADJ 18-20 OMIT HL           | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820470      | FP ADLT ADJ 18-20 OMIT HL           | LEFT                                       |
| 1-3,5,6,8,9,12          | 921466      | ADLT ANGL ADJ F/PLT 12"/13" OMIT HL | PAIR                                       |
| 1-3,5,6,8,9,12          | 921320      | FP ADLT ADJ 14/15 OMIT HL           | PAIR                                       |
| 1-3,5,6,8,9,12          | 921322      | FP ADLT ADJ 16/17 OMIT HL           | PAIR                                       |
| 1-3,5,6,8,9,12          | 921324      | FP ADLT ADJ 18-20 OMIT HL           | PAIR                                       |
| 1-3,5,6,8,9,12          | 820522      | AD ANGL ADJ F/PLT R 12/13" H/L      | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820532      | AD ANGL ADJ F/PLT L 12/13" H/L      | LEFT                                       |
| 1-3,5,6,8-10,11C, 12,14 | 820461      | FP ADLT ADJ 14/15 HL                | RIGHT                                      |
| 1-3,5,6,8-10,11C, 12,14 | 820463      | FP ADLT ADJ 14/15 HL                | LEFT                                       |
| 1-3,5,6,8-10,11C, 12,14 | 820465      | FP ADLT ADJ 16/17 HL                | RIGHT                                      |
| 1-3,5,6,8-10,11C, 12,14 | 820467      | FP ADLT ADJ 16/17 HL                | LEFT                                       |
| 1-3,5,6,8-10,11D, 12,14 | 820469      | FP ADLT ADJ 18-20 HL                | RIGHT                                      |
| 1-3,5,6,8-10,11D, 12,14 | 820471      | FP ADLT ADJ 18-20 HL                | LEFT                                       |
| 1-3,5,6,8-10,11B, 12,14 | 921462      | ADLT ANGL ADJ F/PLATE 12"/13" HL    | PAIR                                       |
| 1-3,5,6,8-10,11C, 12,14 | 921321      | FP ADLT ADJ 14/15 HL                | PAIR                                       |
| 1-3,5,6,8-10,11C, 12,14 | 921323      | FP ADLT ADJ 16/17 HL                | PAIR                                       |
| 1-3,5,6,8-10,11D, 12,14 | 921325      | FP ADLT ADJ 18-20 HL                | PAIR                                       |
| 1-3,6,7A,9-10,14        | 101533      | ADLT ANG ADJ FP D2 HDWR KIT         | FOR DOUBLE D EXT TUBES                     |
| 1-3B,4B,6,7B,9,10,14    | 101534      | ADLT ANG ADJ FP RND HDWR KIT        | FOR ROUND EXT TUBES (Q2 HP ONLY)           |

**NOTE:** \* ORDER ITEM 8 PRIOR TO 8/13/01.


\*\* ORDER ITEM 5 PART 226167 PRIOR TO 8/13/01.

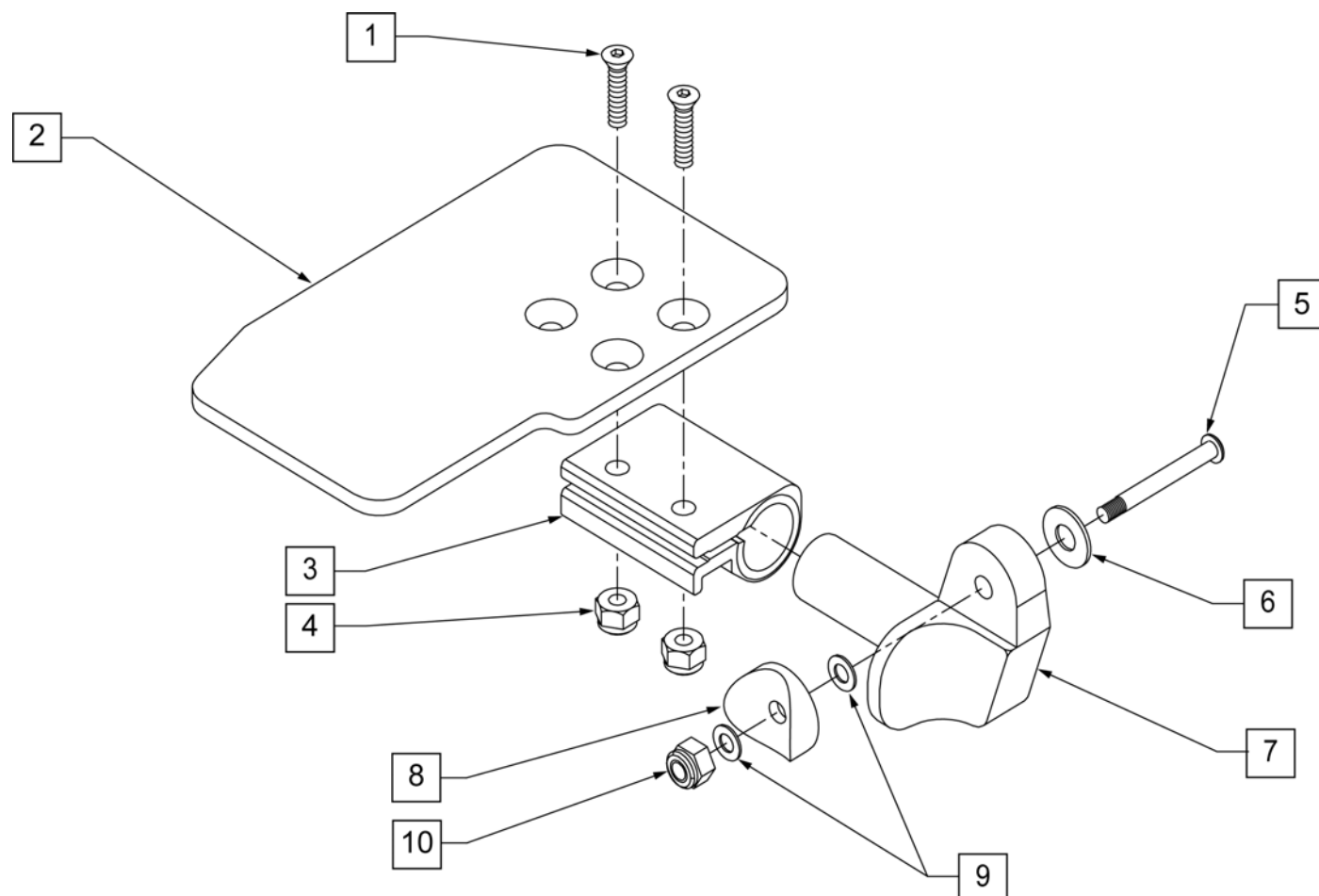

| Pos. | Item Number | Description                    | Remarks |
|------|-------------|--------------------------------|---------|
| 1    | 106978      | 1/4-28 X 7/8 FH C/S            |         |
| 2    | 500635      | KIDS ANGLE ADJ FOOTPLATE 10" R |         |
| 2    | 500636      | KIDS ANGLE ADJ FOOTPLATE 10" L |         |
| 2    | 500637      | KIDS ANGLE ADJ FOOTPLATE 11" R |         |
| 2    | 500638      | KIDS ANGLE ADJ FOOTPLATE 11" L |         |
| 2    | 500639      | KIDS ANGLE ADJ FOOTPLATE 12" R |         |
| 2    | 500640      | KIDS ANGLE ADJ FOOTPLATE 12" L |         |
| 2    | 500641      | KIDS ANGLE ADJ FOOTPLATE 13" R |         |
| 2    | 500642      | KIDS ANGLE ADJ FOOTPLATE 13" L |         |
| 2    | 500643      | KIDS ANGLE ADJ FOOTPLATE 14" R |         |
| 2    | 500644      | KIDS ANGLE ADJ FOOTPLATE 14" L |         |
| 2    | 500645      | KIDS ANGLE ADJ FOOTPLATE 15" R |         |
| 2    | 500646      | KIDS ANGLE ADJ FOOTPLATE 15" L |         |
| 2    | 500647      | KIDS ANGLE ADJ FOOTPLATE 16" R |         |
| 2    | 500648      | KIDS ANGLE ADJ FOOTPLATE 16" L |         |
| 3    | 550350      | 3/4" CLAMP ADJ FOOTPLATE       |         |
| 4    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS    |         |


| Pos.     | Item Number | Description                             | Remarks |
|----------|-------------|-----------------------------------------|---------|
| 5**      | 106962      | 1/4-28 X 1.85 BH C/S BLK ZN             |         |
| 6        | 106945      | 1/4 SAE WASHER                          |         |
| 7        | 541002      | FOOTREST MOUNTING BRACKET - 3/8" RADIUS | RIGHT   |
| 7        | 541003      | FOOTREST MOUNTING BRACKET - 3/8" RADIUS | LEFT    |
| 8*       | 561055      | 3/4" SADDLE 1" OD                       |         |
| 9        | 106944      | 1/4 AN WASHER FLAT THIN                 |         |
| 10       | 106941      | 1/4-28 AUTOMATION NUT                   |         |
| 1-7,9,10 | 828850      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 10" R  | RIGHT   |
| 1-7,9,10 | 828851      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 10" L  | LEFT    |
| 1-7,9,10 | 828852      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 11" R  | RIGHT   |
| 1-7,9,10 | 828853      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 11" L  | LEFT    |
| 1-7,9,10 | 828854      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 12" R  | RIGHT   |
| 1-7,9,10 | 828855      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 12" L  | LEFT    |
| 1-7,9,10 | 828856      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 13" R  | RIGHT   |
| 1-7,9,10 | 828857      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 13" L  | LEFT    |
| 1-7,9,10 | 828858      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 14" R  | RIGHT   |
| 1-7,9,10 | 828859      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 14" L  | LEFT    |
| 1-7,9,10 | 828860      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 15" R  | RIGHT   |
| 1-7,9,10 | 828861      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 15" L  | LEFT    |
| 1-7,9,10 | 828862      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 16" R  | RIGHT   |
| 1-7,9,10 | 828863      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 16" L  | LEFT    |
| 1-7,9,10 | 921280      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 10"    | PAIR    |
| 1-7,9,10 | 921281      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 11"    | PAIR    |
| 1-7,9,10 | 921282      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 12"    | PAIR    |
| 1-7,9,10 | 921283      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 13"    | PAIR    |
| 1-7,9,10 | 921284      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 14"    | PAIR    |
| 1-7,9,10 | 921285      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 15"    | PAIR    |
| 1-7,9,10 | 921286      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 16"    | PAIR    |

NOTE: \* FOR ORDERING INFORMATION, PLEASE SEE "PLATFORM FLIP-UP FOOTREST REFERENCE GUIDE" ATTACHED.

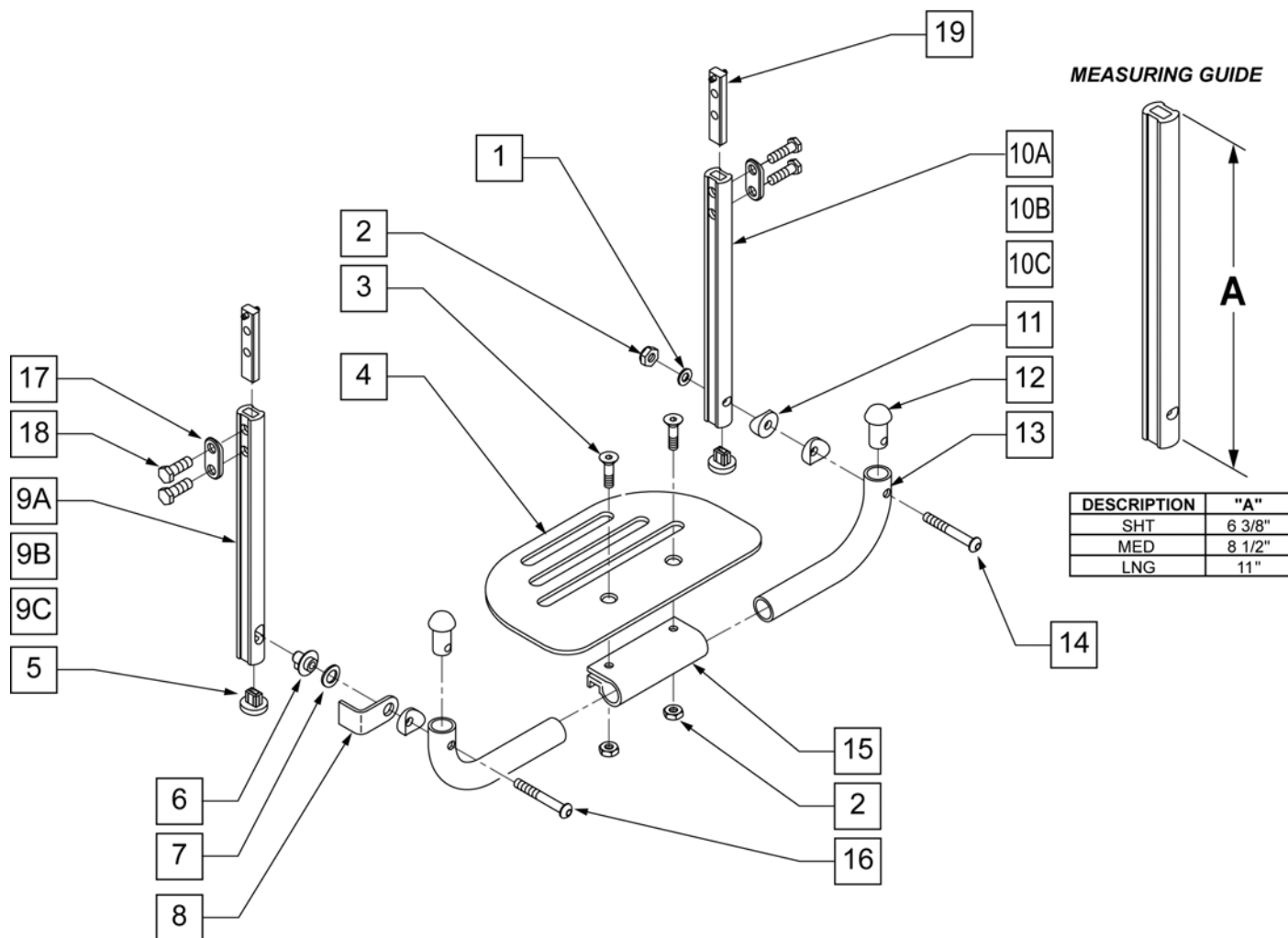


| Pos. | Item Number | Description                 | Remarks |
|------|-------------|-----------------------------|---------|
| 1    | 106944      | 1/4 AN WASHER FLAT THIN     |         |
| 2    | 106941      | 1/4-28 AUTOMATION NUT       |         |
| 3    | 106978      | FHCS 1/4-28 X 7/8 ROHS      |         |
| 4    | 500650      | PLATFORM F/PLT 6" x 5 1/2"  |         |
| 4    | 500651      | PLATFORM F/PLT 8 x 5 1/2"   |         |
| 4    | 500652      | PLATFORM F/PLT 10" x 5 1/2" |         |
| 4    | 500653      | PLATFORM F/PLT 12" x 5 1/2" |         |
| 4    | 500654      | PLATFORM F/PLT 14" x 5 1/2" |         |
| 5    | 113221      | PLUD DOUBLE D EXT TUBE      |         |
| 6    | 540410      | F/UP FOOTREST KEY STUD D2   |         |
| 7    | 262137      | 3/8 WASHER CURVED           |         |
| 8    | 505078      | FOOTREST RETAINING BRACKET  |         |
| 9C   | 550830      | EXT TUBE F/UP KEY ONLY LONG |         |

| Pos.            | Item Number | Description                      | Remarks                    |
|-----------------|-------------|----------------------------------|----------------------------|
| 9A              | 550828      | EXT TUBE F/UP KEY ONLY SHORT     |                            |
| 9B              | 550829      | EXT TUBE F/UP KEY ONLY MED       |                            |
| 10C             | 550827      | EXT TUBE F/UP SLT ONLY LONG      |                            |
| 10A             | 550825      | EXT TUBE F/UP SLT ONLY SHORT     |                            |
| 10B             | 550826      | EXT TUBE F/UP SLT ONLY MED       |                            |
| 11              | 561003      | 3/4" SADDLE THIN                 |                            |
| 12              | 561034      | SHORT DOME PLUG                  |                            |
| 14              | 106964      | 1/4-28 x 2 BH C/S                |                            |
| 15              | 550351      | PLATFORM FOOTPLATE CLAMP         |                            |
| 16              | 226131      | 1/4-28 x 1 1/4 BH C/S SHLDR      |                            |
| 17              | 483717      | HANGER SADDLE                    |                            |
| 18              | 221102      | 1/4-28 x 3/4 HEX C/S             |                            |
| 19              | 500078      | INSERT D2 EXT TUBE               |                            |
| 5,10A, 17-19    | 921742      | EXT TBE F/UP SLT SHT ASSM PCKT   | Pivot side                 |
| 5,10B, 17-19    | 921743      | EXT TBE F/UP SLT MED ASSM PCKT   | Pivot side                 |
| 5,10C, 17-19    | 921744      | EXT TBE F/UP SLT LNG ASSM PCKT   | Pivot side                 |
| 5,9A, 17-19     | 921745      | EXT TBE F/UP KEY SHT ASSM PCKT   | Release side               |
| 5,9B, 17-19     | 921746      | EXT TBE F/UP KEY MED ASSM PCKT   | Release side               |
| 5,9C, 17-19     | 921747      | EXT TBE F/UP KEY LNG ASSM PCKT   | Release side               |
| 5,9A,10A, 17-19 | 921269      | S/A EXT TUBE D2 SHT PLTFRM BLK   | Pivot and Release Ext Assm |
| 5,9B,10B, 17-19 | 921267      | S/A EXT TUBE D2 MED PLTFRM BLK   | Pivot and Release Ext Assm |
| 5,9C,10C, 17-19 | 921268      | S/A EXT TUBE D2 LNG PLTFRM BLK   | Pivot and Release Ext Assm |
| 1-4,6-8, 11-16  | 900304      | PLATFM F/P F/PLT BLK 10" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900305      | PLATFM F/P F/PLT BLK 11" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900306      | PLATFM F/P F/PLT BLK 12" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900307      | PLATFM F/P F/PLT BLK 13" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900308      | PLATFM F/P F/PLT BLK 14" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900309      | PLATFM F/P F/PLT BLK 15" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900310      | PLATFM F/P F/PLT BLK 16" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900311      | PLATFM F/P F/PLT BLK 17" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900312      | PLATFM F/P F/PLT BLK 18" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900313      | PLATFM F/P F/PLT BLK 19" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900314      | PLATFM F/P F/PLT BLK 20" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900315      | PLATFM F/P F/PLT BLK 21" ASSM D2 |                            |
| 1-4,6-8, 11-16  | 900316      | PLATFM F/P F/PLT BLK 22" ASSM D2 |                            |

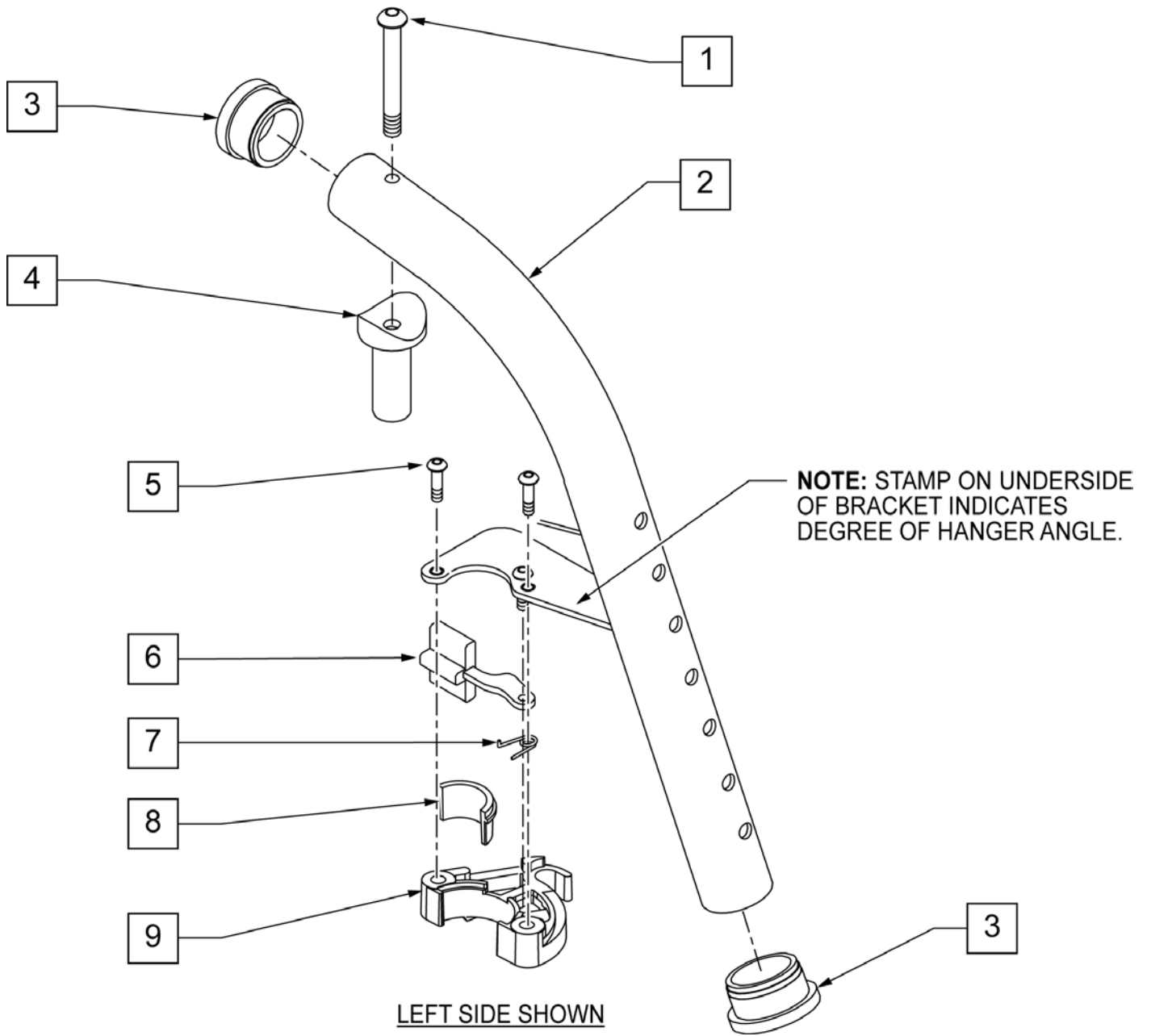


| Pos. | Item Number | Description                 | Remarks |
|------|-------------|-----------------------------|---------|
| 1    | 226217      | 5/16-18 x 2 BH C/S BLK      |         |
| 3    | 561002      | 1" INSERT PLUG .083         |         |
| 4    | 561197      | PLASTIC PIVOT SADDLE        |         |
| 5    | 226057      | 10/24 x .725 BTTN HD SOC BZ |         |
| 6    | 561079      | S/A LATCH LEVER             |         |
| 7    | 321136      | S/A SPRING L RED            | LEFT    |
| 7    | 321137      | S/A SPRING R CLEAR          | RIGHT   |
| 8,9  | 881500      | LATCH BLOCK ASSM R          | RIGHT   |
| 8,9  | 881501      | LATCH BLOCK ASSM L          | LEFT    |
| 1-9  | 641142S     | HANGER 70D R PNTD S/A F/MT  | RIGHT   |
| 1-9  | 641143S     | HANGER 70D L PNTD S/A F/MT  | LEFT    |
| 5-9  | 881521      | S/A LATCH KIT CASTING       | RIGHT   |
| 5-9  | 881522      | S/A LATCH KIT CASTING       | LEFT    |

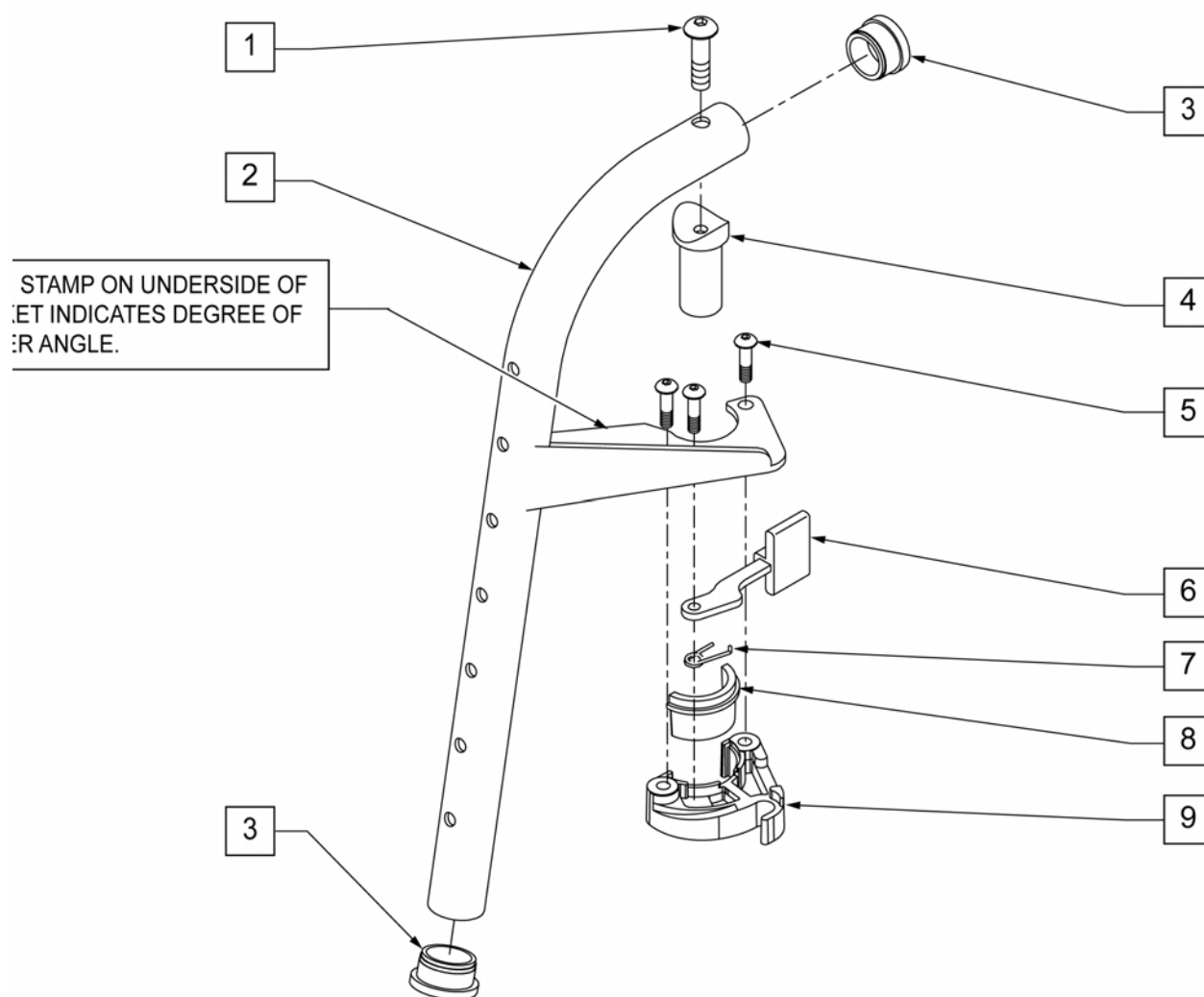

| Pos. | Item Number | Description                 | Remarks |
|------|-------------|-----------------------------|---------|
| 1    | 226217      | 5/16-18 x 2 BH C/S BLK      |         |
| 3    | 561002      | 1" INSERT PLUG .083         |         |
| 4    | 561197      | PLASTIC PIVOT SADDLE        |         |
| 5    | 226057      | 10/24 x .725 BTTN HD SOC BZ |         |
| 6    | 561079      | S/A LATCH LEVER             |         |
| 7    | 321136      | S/A SPRING L RED            | LEFT    |
| 7    | 321137      | S/A SPRING R CLEAR          | RIGHT   |
| 8,9  | 881500      | LATCH BLOCK ASSM R          | RIGHT   |
| 8,9  | 881501      | LATCH BLOCK ASSM L          | LEFT    |
| 1-9  | 641144S     | HANGER 80D R PNTD S/A F/MT  | RIGHT   |
| 1-9  | 641145S     | HANGER 80D L PNTD S/A F/MT  | LEFT    |
| 5-9  | 881521      | S/A LATCH KIT CASTING       | RIGHT   |
| 5-9  | 881522      | S/A LATCH KIT CASTING       | LEFT    |




**NOTE:** DISCONTINUED ON LX/LXI 10/24/2008 -PARTS ONLY FOR CHAIRS BUILT PRIOR TO SERIAL NUMBER LX-034330.

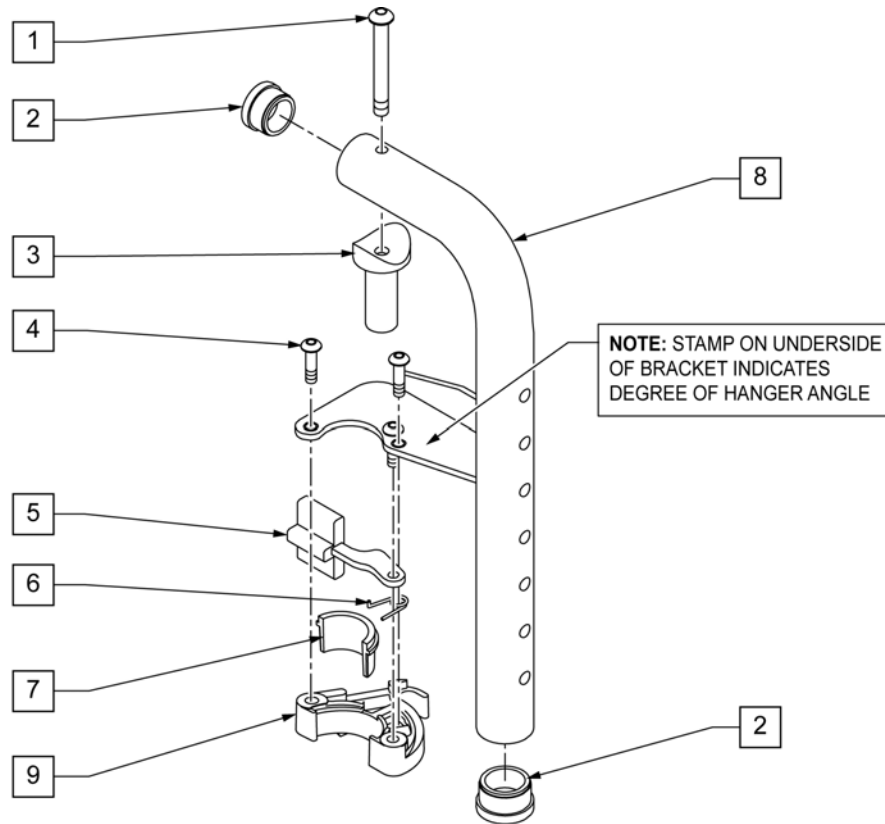


| Pos. | Item Number | Description                 | Remarks |
|------|-------------|-----------------------------|---------|
| 1    | 226217      | 5/16-18 x 2 BH C/S BLK      |         |
| 2    | 561002      | 1" INSERT PLUG .083         |         |
| 3    | 561197      | PLASTIC PIVOT SADDLE        |         |
| 4    | 226057      | 10/24 x .725 BTTN HD SOC BZ |         |
| 5    | 561079      | S/A LATCH LEVER             |         |
| 6    | 321136      | S/A SPRING L RED            | LEFT    |
| 6    | 321137      | S/A SPRING R CLEAR          | RIGHT   |
| 1,3  | 880545      | PIVOT SADDLE .125 W/SCREW   |         |
| 7,9  | 881500      | LATCH BLOCK ASSM R          | RIGHT   |
| 7,9  | 881501      | LATCH BLOCK ASSM L          | LEFT    |
| 1-9  | 641556S     | HANGER 90D R PNTD S/A       | RIGHT   |
| 1-9  | 641557S     | HANGER 90D L PNTD S/A       | LEFT    |
| 5-9  | 881521      | S/A LATCH KIT CASTING       | RIGHT   |
| 5-9  | 881522      | S/A LATCH KIT CASTING       | LEFT    |

**NOTE:** \* ORDER ITEM 16 WHEN NOT USING FOOTREST EXTENSION.

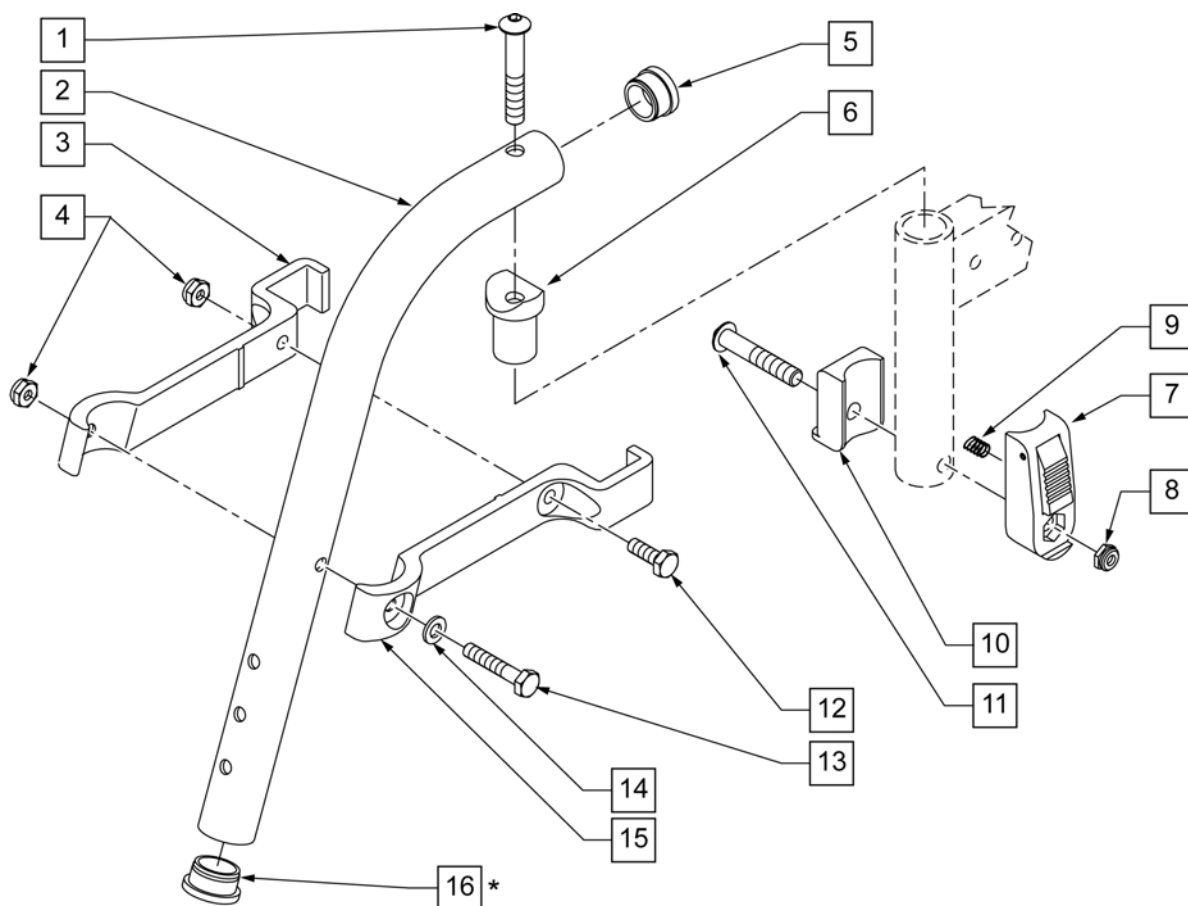


| Pos.      | Item Number | Description                       | Remarks                                 |
|-----------|-------------|-----------------------------------|-----------------------------------------|
| 1         | 106983      | BHCS 5/16 -18 x 2 SHLDR ROHS      |                                         |
| 3         | 550152      | T45 HANGER BRACKET 70ø R          |                                         |
| 4         | 106935      | NUT NYLOCK THIN 1/4 -28 ROHS      |                                         |
| 5         | 561002      | 1" INSERT PLUG .083               |                                         |
| 6         | 561197      | PLASTIC PIVOT SADDLE              |                                         |
| 7         | 880053      | FOOTREST BLOCK T45                |                                         |
| 8         | 202117      | 5/16-18 NYLOCK NUT THICK BLK      |                                         |
| 9         | 321141      | .650 x 6 COIL SPRING              |                                         |
| 10        | 560684      | T45 FOOTREST BLOCK RECESSED       |                                         |
| 11        | 107029      | BHCS 5/16 -18 x 1 5/8 SHLD ROHS   |                                         |
| 12        | 107014      | HHCS 1/4 -28 x 3/4 ROHS           |                                         |
| 13        | 106971      | HHCS 1/4 -28 x 1 3/8 ROHS         |                                         |
| 14        | 106943      | WASHER FLAT AN 1/4 ROHS           |                                         |
| 15        | 550153      | T45 HANGER BRACKET 70ø L          |                                         |
| 16        | 561002      | 1" INSERT PLUG .083               | ORDER WHEN NOT USING FOOTREST EXTENSION |
| 7-11      | 920720      | FOOTREST BLOCK ASSEMBLY W/BOLT HD |                                         |
| 1-6,12-15 | 641210S     | HANGER 70D PNTD R HD              |                                         |
| 1-6,12-15 | 641211S     | HANGER 70D PNTD L HD              |                                         |

**NOTE:** \* ORDER ITEM 16 WHEN NOT USING FOOTREST EXTENSION.

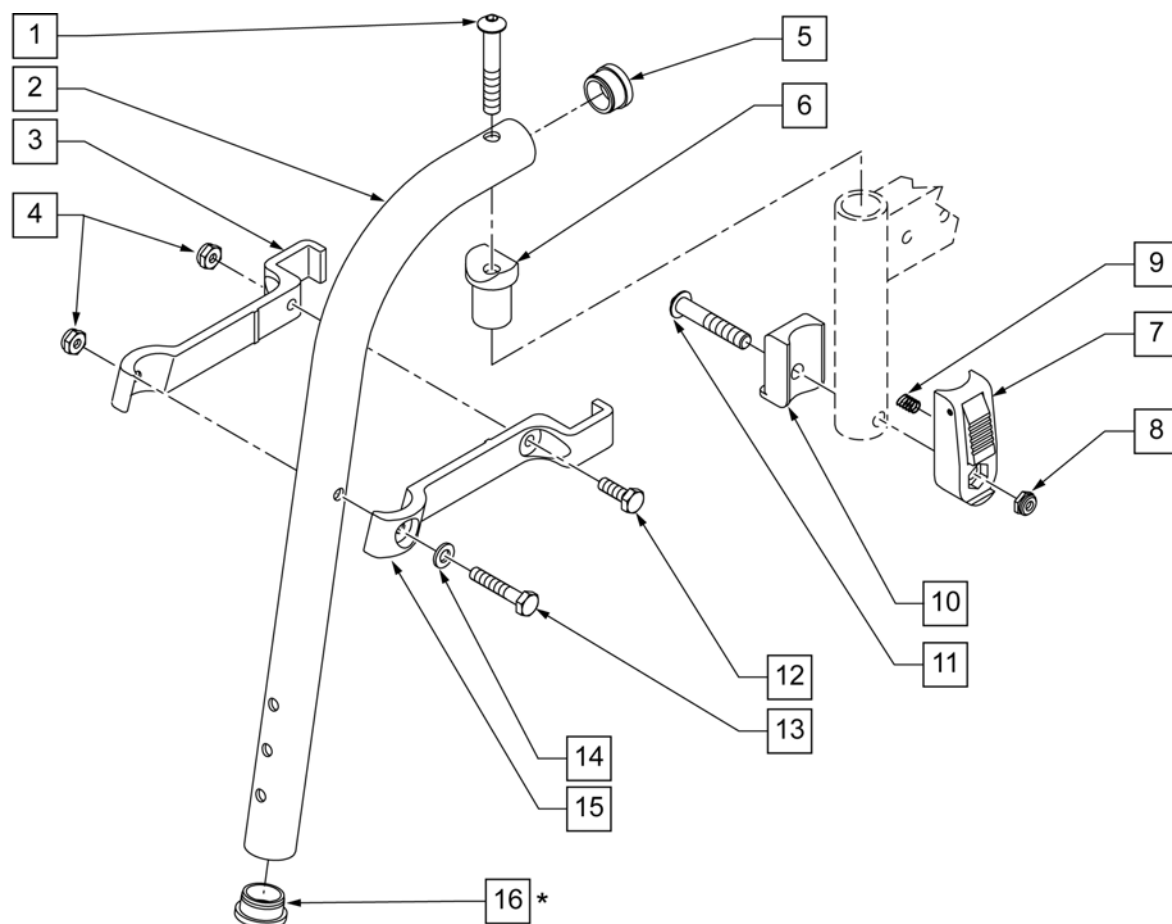


| Pos.      | Item Number | Description                       | Remarks                                 |
|-----------|-------------|-----------------------------------|-----------------------------------------|
| 1         | 106983      | BHCS 5/16 -18 x 2 SHLDR ROHS      |                                         |
| 3         | 550154      | T45 HANGER BRACKET 80ø R          |                                         |
| 4         | 106935      | NUT NYLOCK THIN 1/4 -28 ROHS      |                                         |
| 5         | 561002      | 1" INSERT PLUG .083               |                                         |
| 6         | 561197      | PLASTIC PIVOT SADDLE              |                                         |
| 7         | 880053      | FOOTREST BLOCK T45                |                                         |
| 8         | 202117      | 5/16-18 NYLOCK NUT THICK BLK      |                                         |
| 9         | 321141      | .650L x 6 COIL SPRING             |                                         |
| 10        | 560684      | T45 FOOTREST BLOCK RECESSED       |                                         |
| 11        | 107029      | BHCS 5/16 -18 x 1 5/8 SHLD ROHS   |                                         |
| 12        | 107014      | HHCS 1/4 -28 x 3/4 ROHS           |                                         |
| 13        | 106971      | HHCS 1/4 -28 x 1 3/8 ROHS         |                                         |
| 14        | 106943      | WASHER FLAT AN 1/4 ROHS           |                                         |
| 15        | 550155      | T45 HANGER BRACKET 80ø L          |                                         |
| 16        | 561002      | 1" INSERT PLUG .083               | ORDER WHEN NOT USING FOOTREST EXTENSION |
| 7-11      | 920720      | FOOTREST BLOCK ASSEMBLY W/BOLT HD |                                         |
| 1-6,12-15 | 641200S     | HANGER 80D PNTD R HD              |                                         |
| 1-6,12-15 | 641201S     | HANGER 80D PNTD L HD              |                                         |



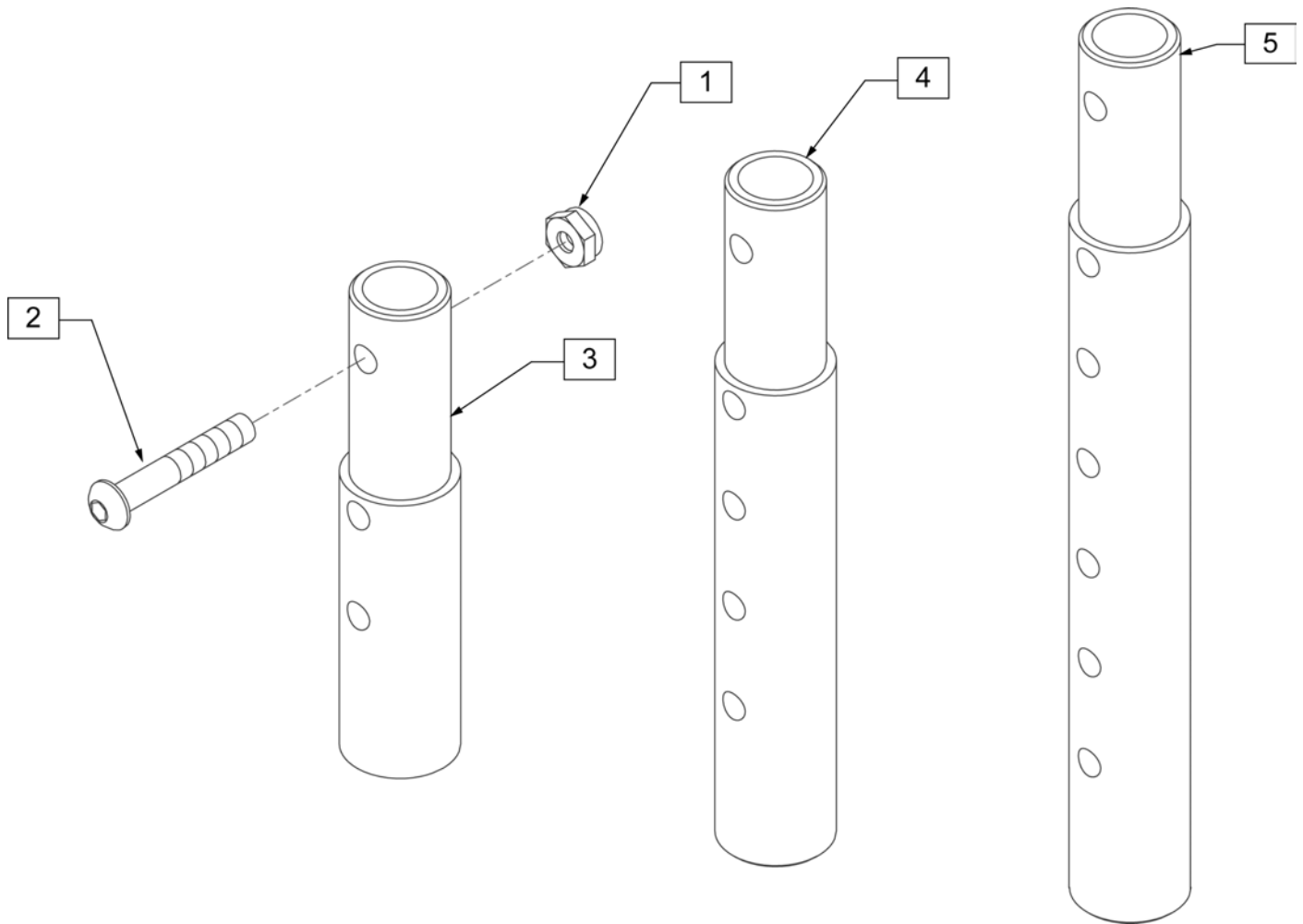


| Pos.    | Item Number | Description                             | Remarks       |
|---------|-------------|-----------------------------------------|---------------|
| 1       | 106935      | NUT NYLOCK THIN 1/4 -28 ROHS            |               |
| 2       | 106961      | BHCS 1/4 -28 x 1 1/4 ROHS               |               |
| 1-3     | 111505      | 2" FOOTREST EXTENSION TUBE W/ HDWR ASSM | 2" EXT (PAIR  |
| 1, 2, 4 | 111506      | 4" FOOTREST EXTENSION TUBE W/ HDWR ASSM | 4" EXT (PAIR) |
| 1, 2, 5 | 111507      | 6" FOOTREST EXTENSION TUBE W/ HDWR ASSM | 6" EXT (PAIR) |

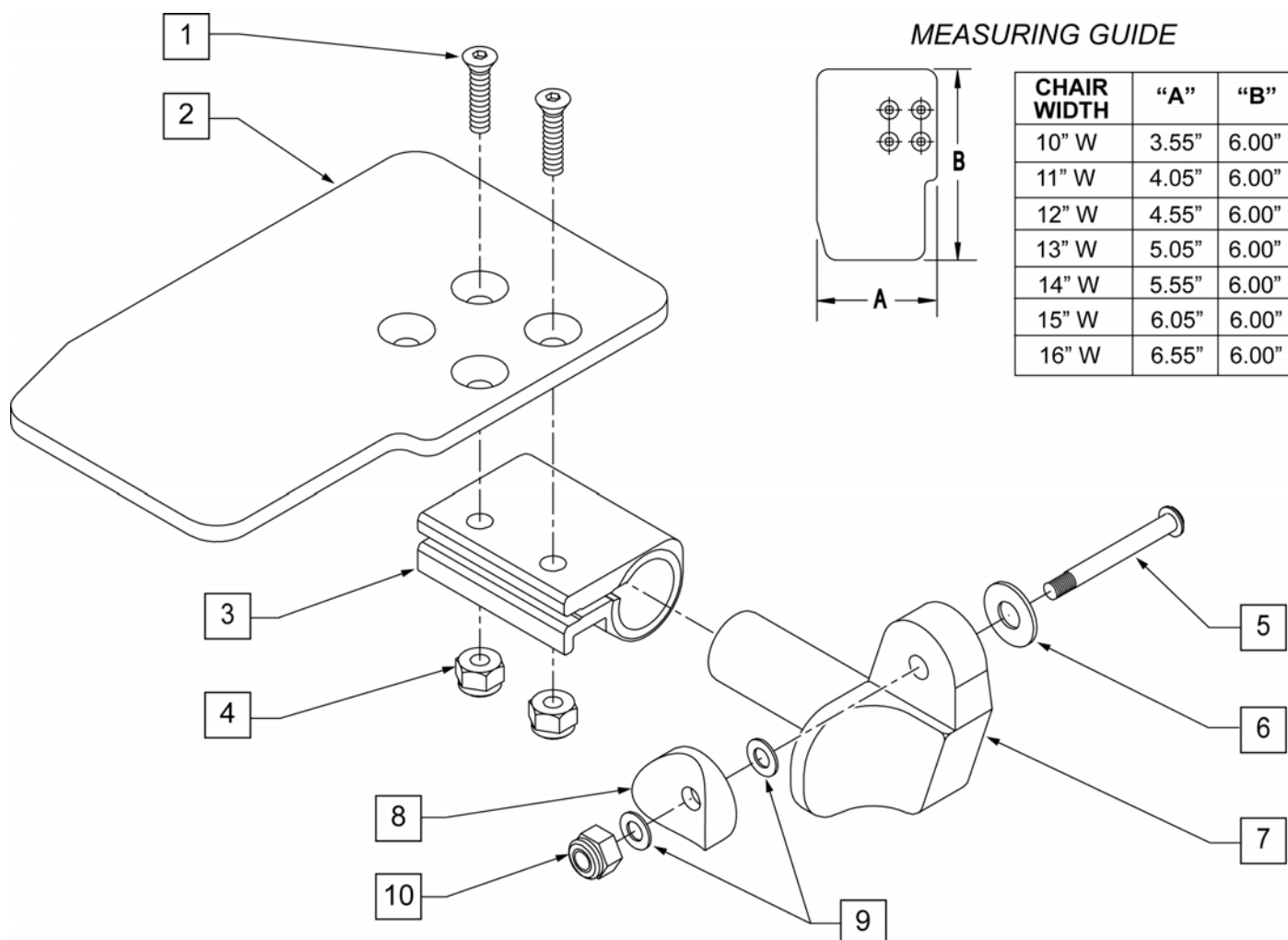

| Pos. | Item Number | Description                 | Remarks | Pos. | Item Number | Description                | Remarks |
|------|-------------|-----------------------------|---------|------|-------------|----------------------------|---------|
| 1    | 106978      | FHCS 1/4-28 X 7/8 ROHS      |         | 7    | 541001      | FTRST MOUNTING BRKT 1 RAD  | LEFT    |
| 2    | 500635      | KIDS ANGL ADJ F/PLT 10      | RIGHT   | 8    | 561004      | 1 SADDLE 1/4 HOLE          |         |
| 2    | 500636      | KIDS ANGL ADJ F/PLT 10      | LEFT    | 9    | 106943      | WASHER FLAT THIN AN 1/4    |         |
| 2    | 500637      | KIDS ANGL ADJ F/PLT 11      | RIGHT   | 10   | 106941      | NUT AUTOMATION 1/4-28 ROHS |         |
| 2    | 500638      | KIDS ANGL ADJ F/PLT 11      | LEFT    | 1-10 | 828821      | KIDS ANG ADJ F/PLT ASSM 10 | RIGHT   |
| 2    | 500639      | KIDS ANGL ADJ F/PLT 12      | RIGHT   | 1-10 | 828822      | KIDS ANG ADJ F/PLT ASSM 10 | LEFT    |
| 2    | 500640      | KIDS ANGL ADJ F/PLT 12      | LEFT    | 1-10 | 828823      | KIDS ANG ADJ F/PLT ASSM 11 | RIGHT   |
| 2    | 500641      | KIDS ANGL ADJ F/PLT 13      | RIGHT   | 1-10 | 828824      | KIDS ANG ADJ F/PLT ASSM 11 | LEFT    |
| 2    | 500642      | KIDS ANGL ADJ F/PLT 13      | LEFT    | 1-10 | 828825      | KIDS ANG ADJ F/PLT ASSM 12 | RIGHT   |
| 2    | 500643      | KIDS ANGL ADJ F/PLT 14      | RIGHT   | 1-10 | 828826      | KIDS ANG ADJ F/PLT ASSM 12 | LEFT    |
| 2    | 500644      | KIDS ANGL ADJ F/PLT 14      | LEFT    | 1-10 | 828827      | KIDS ANG ADJ F/PLT ASSM 13 | RIGHT   |
| 2    | 500645      | KIDS ANGL ADJ F/PLT 15      | RIGHT   | 1-10 | 828828      | KIDS ANG ADJ F/PLT ASSM 13 | LEFT    |
| 2    | 500646      | KIDS ANGL ADJ F/PLT 15      | LEFT    | 1-10 | 828829      | KIDS ANG ADJ F/PLT ASSM 14 | RIGHT   |
| 2    | 500647      | KIDS ANGL ADJ F/PLT 16      | RIGHT   | 1-10 | 828830      | KIDS ANG ADJ F/PLT ASSM 14 | LEFT    |
| 2    | 500648      | KIDS ANGL ADJ F/PLT 16      | LEFT    | 1-10 | 828831      | KIDS ANG ADJ F/PLT ASSM 15 | RIGHT   |
| 3    | 550350      | 3/4 CLAMP ADJ FOOTPLATE     |         | 1-10 | 828832      | KIDS ANG ADJ F/PLT ASSM 15 | LEFT    |
| 4    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS |         | 1-10 | 828833      | KIDS ANG ADJ F/PLT ASSM 16 | RIGHT   |
| 5    | 226147      | 1/4-28 X 1.85 BH C/S BLK ZN |         | 1-10 | 828834      | KIDS ANG ADJ F/PLT ASSM 16 | LEFT    |
| 6    | 106945      | WASHER FLAT SAE 1/4 ROHS    |         | 1-10 | 921226      | KIDS ANG ADJ F/PLT ASSM 10 | PAIR    |
| 7    | 541000      | FTRST MOUNTING BRKT 1 RAD   | RIGHT   | 1-10 | 921227      | KIDS ANG ADJ F/PLT ASSM 11 | PAIR    |


| Pos. | Item Number | Description                | Remarks | Pos. | Item Number | Description                | Remarks |
|------|-------------|----------------------------|---------|------|-------------|----------------------------|---------|
| 1-10 | 921228      | KIDS ANG ADJ F/PLT ASSM 12 | PAIR    | 1-10 | 921231      | KIDS ANG ADJ F/PLT ASSM 15 | PAIR    |
| 1-10 | 921229      | KIDS ANG ADJ F/PLT ASSM 13 | PAIR    | 1-10 | 921232      | KIDS ANG ADJ F/PLT ASSM 16 | PAIR    |
| 1-10 | 921230      | KIDS ANG ADJ F/PLT ASSM 14 | PAIR    |      |             |                            |         |

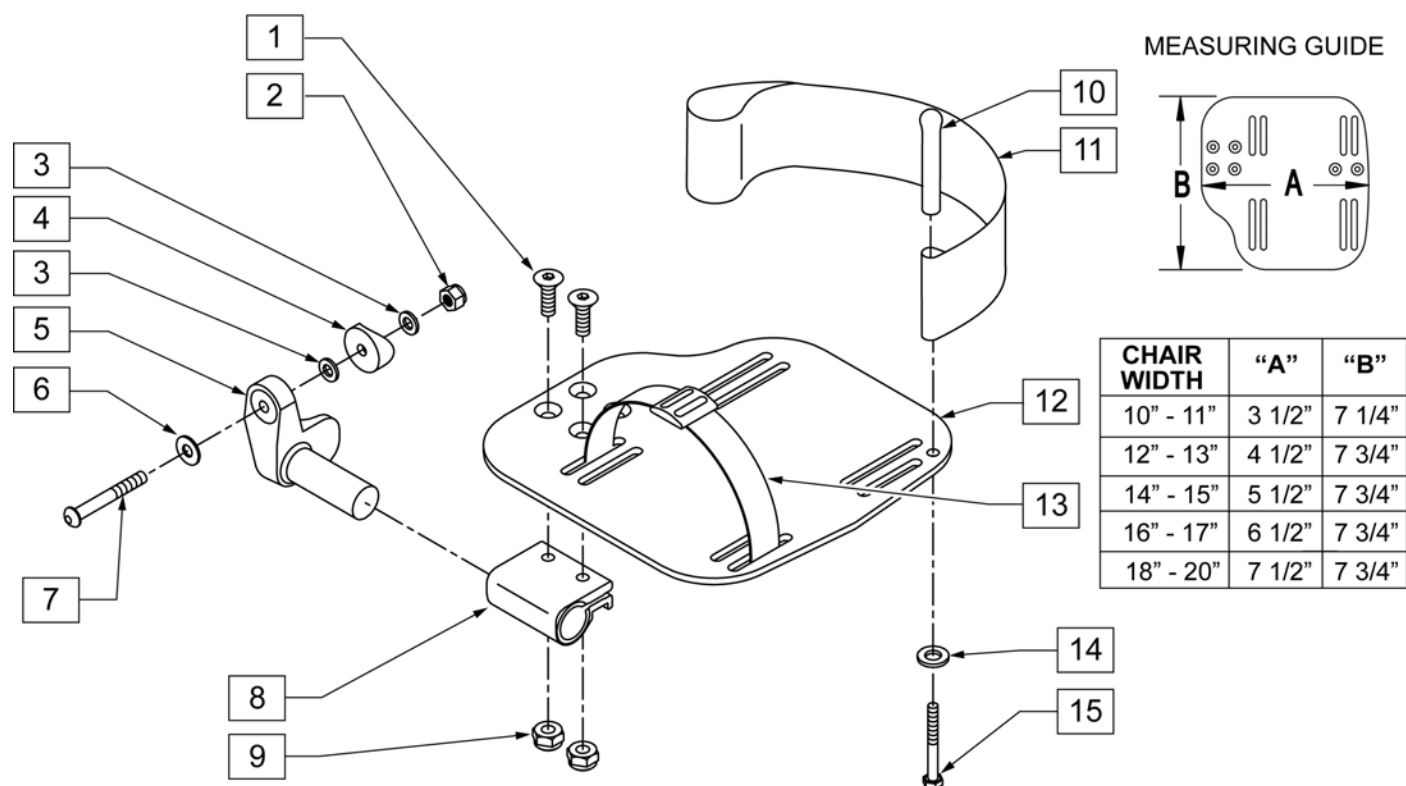

| Pos. | Item Number | Description                         | Remarks                                    |
|------|-------------|-------------------------------------|--------------------------------------------|
| 1    | 106978      | FHCS 1/4-28 X 7/8 ROHS              |                                            |
| 2    | 106941      | NUT AUTOMATION 1/4-28 ROHS          |                                            |
| 3    | 106943      | WASHER FLAT AN 1/4 ROHS             |                                            |
| 4    | 561004      | 1" SADDLE 1/4" HOLE                 |                                            |
| 5    | 541000      | FOOTREST MOUNTING BRKT-1" RAD       | RIGHT                                      |
| 5    | 541001      | FOOTREST MOUNTING BRKT-1" RAD       | LEFT                                       |
| 6    | 106945      | WASHER FLAT SAE 1/4 ROHS            |                                            |
| 7    | 106964      | BHCS 1/4-28 X 2                     |                                            |
| 8    | 550350      | 3/4" CLAMP ADJ FOOTPLATE            |                                            |
| 9    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS         |                                            |
| 10   | 561040      | HEEL LOOP PEG                       |                                            |
| 11   | 741153      | HEEL LOOP 18-20 CMPST/14-15 ANG ADJ | USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS |
| 11   | 741253      | HEEL LOOP ADLT ADJ F/PLT 18-20      | USED W/ 18 -20" ADULT ANG ADJ FTPLTS       |
| 12   | 500656      | ADULT ANGL ADJ F/PLT 10/11          | RIGHT                                      |
| 12   | 500657      | ADULT ANGL ADJ F/PLT 10/11          | LEFT                                       |
| 12   | 500658      | ADULT ANGL ADJ F/PLT 12/13          | RIGHT                                      |
| 12   | 500659      | ADULT ANGL ADJ F/PLT 12/13          | LEFT                                       |
| 12   | 500660      | ADULT ANGL ADJ F/PLT 14/15          | RIGHT                                      |
| 12   | 500661      | ADULT ANGL ADJ F/PLT 14/15          | LEFT                                       |
| 12   | 500662      | ADULT ANGL ADJ F/PLT 16/17          | RIGHT                                      |
| 12   | 500663      | ADULT ANGL ADJ F/PLT 16/17          | LEFT                                       |
| 12   | 500664      | ADULT ANGL ADJ F/PLT 18-20          | RIGHT                                      |
| 12   | 500665      | ADULT ANGL ADJ F/PLT 18-20          | LEFT                                       |
| 13   | 740260      | TOE-ANKLE STRAP ADJ ADULT FOOTPLATE |                                            |
| 14   | 106943      | WASHER FLAT AN 1/4 ROHS             |                                            |




| Pos.       | Item Number | Description                       | Remarks |
|------------|-------------|-----------------------------------|---------|
| 15         | 106958      | HHCS 1/4-20 X 1 ROHS              |         |
| 1-9,12     | 820544      | FP ADULT ADJ 90D 10/11 R          | RIGHT   |
| 1-9,12     | 820554      | FP ADULT ADJ 90D 10/11 L          | LEFT    |
| 1-9,12     | 820546      | FP ADULT ADJ 90D 12/13 R          | RIGHT   |
| 1-9,12     | 820556      | FP ADULT ADJ 90D 12/13 L          | LEFT    |
| 1-9,12     | 820472      | FP ADULT ADJ 90D 14/15 R          | RIGHT   |
| 1-9,12     | 820473      | FP ADULT ADJ 90D 14/15 L          | LEFT    |
| 1-9,12     | 820474      | FP ADULT ADJ 90D 16/17 R          | RIGHT   |
| 1-9,12     | 820475      | FP ADULT ADJ 90D 16/17 L          | LEFT    |
| 1-9,12     | 820476      | FP ADULT ADJ 90D 18-20 R          | RIGHT   |
| 1-9,12     | 820477      | FP ADULT ADJ 90D 18-20 L          | LEFT    |
| 1-9,12     | 921474      | ADULT ANGL ADJ 90D 10/11 OMIT H/L | PAIR    |
| 1-9,12     | 921476      | ADULT ANGL ADJ 90D 12/13 OMIT H/L | PAIR    |
| 1-9,12     | 921330      | ADULT ANGL ADJ 90D 14/15 OMIT H/L | PAIR    |
| 1-9,12     | 921331      | ADULT ANGL ADJ 90D 16/17 OMIT H/L | PAIR    |
| 1-9,12     | 921332      | ADULT ANGL ADJ 90D 18-20 OMIT H/L | PAIR    |
| 1-12,14,15 | 820540      | F/PLT ADULT ADJ 90D 10/11 R W/ HL | RIGHT   |
| 1-12,14,15 | 820550      | F/PLT ADULT ADJ 90D 10/11 L W/ HL | LEFT    |
| 1-12,14,15 | 820542      | F/PLT ADULT ADJ 90D 12/13 R W/ HL | RIGHT   |
| 1-12,14,15 | 820552      | F/PLT ADULT ADJ 90D 12/13 L W/ HL | LEFT    |
| 1-12,14,15 | 820478      | F/PLT ADULT ADJ 90D 14/15 R W/ HL | RIGHT   |
| 1-12,14,15 | 820479      | F/PLT ADULT ADJ 90D 14/15 L W/ HL | LEFT    |
| 1-12,14,15 | 820480      | F/PLT ADULT ADJ 90D 16/17 R W/ HL | RIGHT   |
| 1-12,14,15 | 820481      | F/PLT ADULT ADJ 90D 16/17 L W/ HL | LEFT    |
| 1-12,14,15 | 820482      | F/PLT ADULT ADJ 90D 18-20 R W/ HL | RIGHT   |
| 1-12,14,15 | 820483      | F/PLT ADULT ADJ 90D 18-20 L W/ HL | LEFT    |
| 1-12,14,15 | 921470      | F/PLT ADULT ADJ 90D 10/11 W/ HL   | PAIR    |
| 1-12,14,15 | 921472      | F/PLT ADULT ADJ 90D 12/13 W/ HL   | PAIR    |
| 1-12,14,15 | 921333      | F/PLT ADULT ADJ 90D 14/15 W/ HL   | PAIR    |
| 1-12,14,15 | 921334      | F/PLT ADULT ADJ 90D 16/17 W/ HL   | PAIR    |
| 1-12,14,15 | 921335      | F/PLT ADULT ADJ 90D 18-20 W/ HL   | PAIR    |

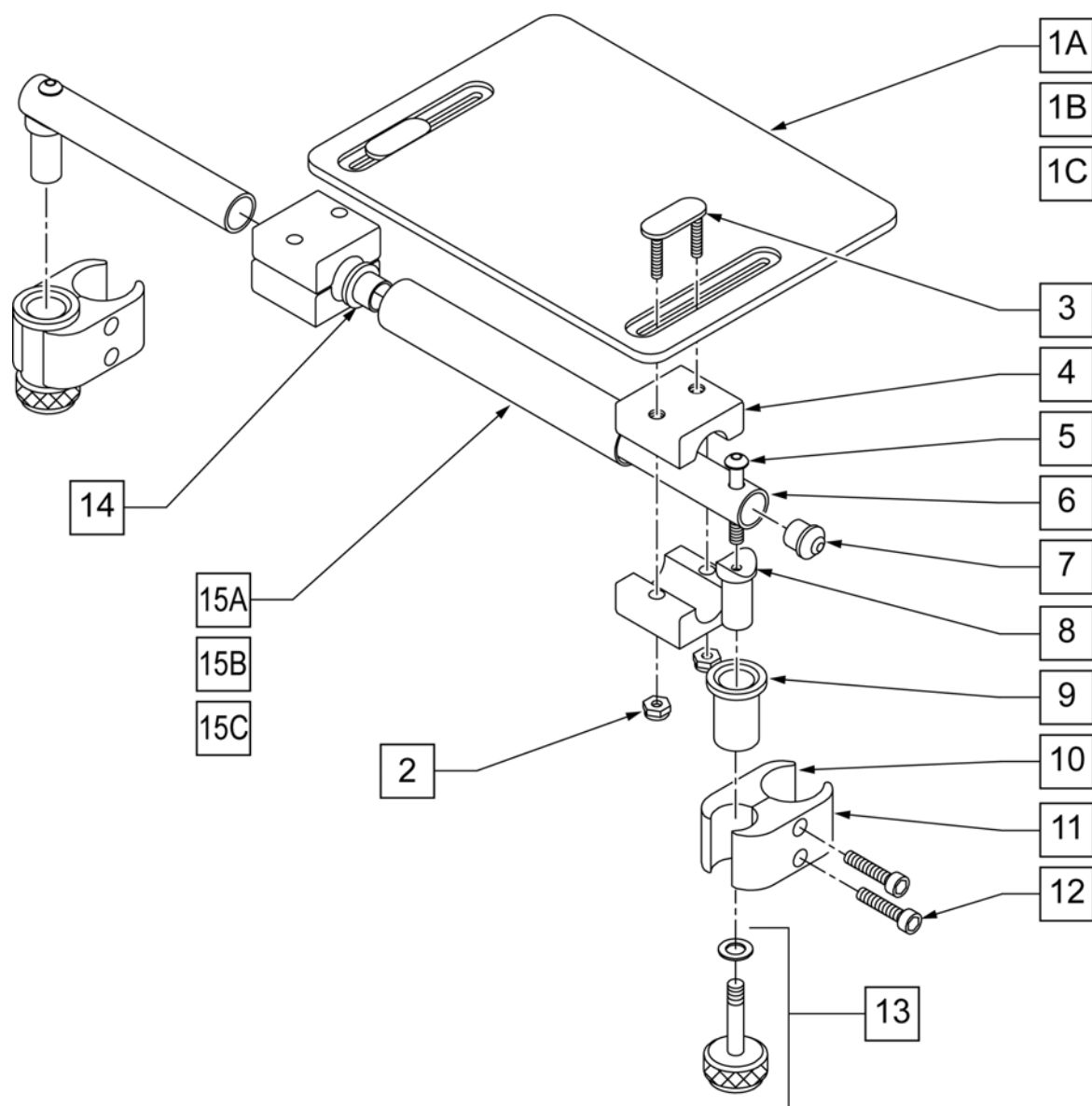


| Pos. | Item Number | Description                               | Remarks      |
|------|-------------|-------------------------------------------|--------------|
| 1A   | 562610      | SLD FOOTPLATE ONLY 8" x 8" (10"-12" W)    | 10 -12" WIDE |
| 1B   | 562612      | SLD FOOTPLATE ONLY 10" x 8" (13"-14" W)   | 13 -14" WIDE |
| 1C   | 500709      | SLD FT PLT ALUM ONLY 12" x 8" (15"-16" W) | 15 -16" WIDE |
| 2    | 106933      | NUT NYLOCK THIN 1/4-20 ROHS               |              |
| 3    | 330120      | SLD FT/ANGLE ADJ DOUBLE T BOLT            |              |
| 4    | 550444      | SOLID FOOT/ANG ADJ CLAMP BLK              |              |
| 5    | 107028      | BHCS 1/4-28 X 1-1/2 ROHS                  |              |
| 7    | 561034      | SHORT DOME PLUG                           |              |
| 8    | 483236      | SOLID FOOT INSERT SADDLE BLK              |              |
| 10   | 550658      | CLAMP 1 1/4" THREADED BLK                 |              |
| 11   | 550657      | CLAMP 1 1/4" RECESS FULL BLK              |              |
| 12   | 107032      | SHCS 1/4-28 X 1 ROHS                      |              |
| 9,13 | 880559      | SOLID FOOT LOCK KNOB ASSEMBLY             |              |

| Pos.         | Item Number | Description                                   | Remarks      |
|--------------|-------------|-----------------------------------------------|--------------|
| 14           | 560402      | 3/4" ID NY-LINER                              |              |
| 15A          | 521621      | SLD FOOT SUPPORT TUBE BLK 10"-12"             | 10 -12" WIDE |
| 15B          | 521622      | SLD FOOT SUPPORT TUBE BLK 13"-14"             | 13 -14" WIDE |
| 15C          | 521624      | SLD FOOT SUPPORT TUBE BLK 15"-16"             | 15 -16" WIDE |
| 5-8          | 880183      | SLD FT PLT PIVOT TUBE SUB-ASSM                |              |
| 1A,2-14, 15A | 921801      | SLD FT PLT ASSEMBLY 8" x 8" (10"-12" W)       | 10 -12" WIDE |
| 1B,2-14, 15B | 921803      | SOLID FOOTPLATE ASSEMBLY 10" x 8" (13"-14" W) | 13 -14" WIDE |
| 1C,2-14, 15C | 921805      | SOLID FOOTPLATE ASSEMBLY 12" x 8" (15"-16" W) | 15 -16" WIDE |

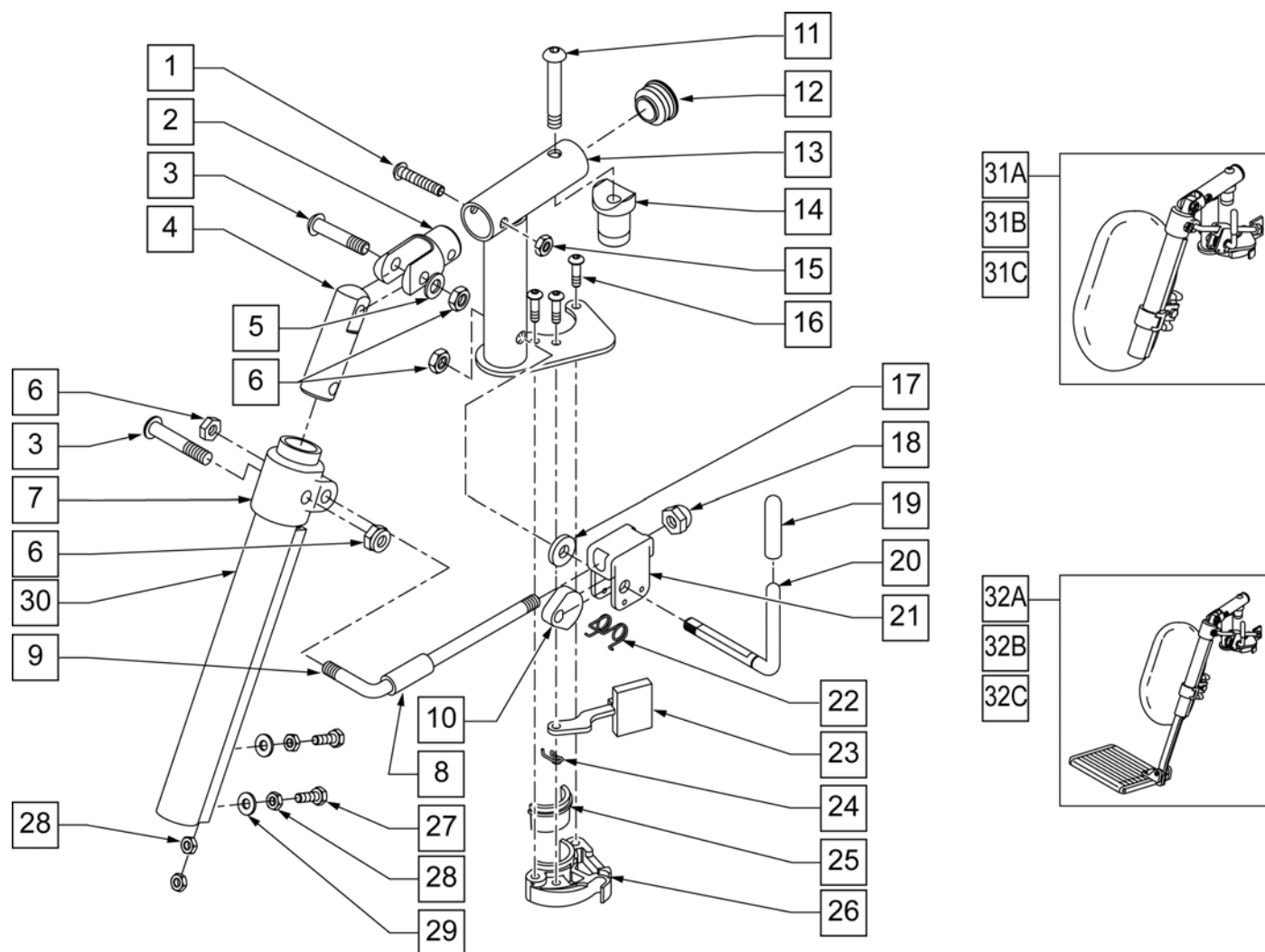


| Pos.  | Item Number | Description                     | Remarks |
|-------|-------------|---------------------------------|---------|
| 1     | 106961      | 1/4-28 X 1 1/4 BH C/S           |         |
| 2     | 505281      | ELR CLEVIS                      |         |
| 3     | 107037      | BHCS 5/16-24 X 1-1/2 SHLD ROHS  |         |
| 4     | 540252      | ELR PIVOT PRS                   |         |
| 5     | 106946      | WASHER FLAT AN 5/16 ROHS        |         |
| 6     | 106939      | NUT NYLOCK THIN 5/16-24 ROHS    |         |
| 7     | 560420      | ELR UPPER PIVOT PLST            |         |
| 8     | 560784      | ELR ADJUSTMENT STOP             |         |
| 9     | 481117      | ELR LOCK ROD                    |         |
| 10    | 481119      | ELR CAM LOCKING POWDERED METAL  |         |
| 11    | 106982      | BHCS 5/16 -18 x 1 3/4 SHLD ROHS |         |
| 12    | 561062      | 1" INSERT PLUG .065 RIBS        |         |
| 14    | 122729      | ELR ALUM PIVOT SADDLE           |         |
| 15    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS     |         |
| 16    | 107035      | BHCS 10-24 X .725 PL ROHS       |         |
| 17    | 561059      | 3/4 OD x .318 ID WASHER         |         |
| 18    | 202109      | 5/16-24 ACORN NUT HEX           |         |
| 19    | 561049      | ELR RELEASE ROD COVER           |         |
| 19,20 | 882410S     | ELR ACTUATION ARM ASSEMBLY      |         |

| Pos.            | Item Number | Description                     | Remarks |
|-----------------|-------------|---------------------------------|---------|
| 21              | 880990      | ELR LOCK HOUSING W/PIN          |         |
| 22              | 321106      | ELR TORSION SPRING              |         |
| 23              | 561079      | S/A LATCH LEVER                 |         |
| 24              | 321136      | S/A SPRING L RED                | LEFT    |
| 24              | 321137      | S/A SPRING R CLEAR              | RIGHT   |
| 27              | 107014      | 1/4-28 x 3/4 HEX C/S            |         |
| 28              | 107249      | NUT HEX THIN 1/4-28 ROHS        |         |
| 29              | 106944      | WASHER FLAT THIN AN 1/4 ROHS    |         |
| 30              | 550722S     | ELR ADJUSTMENT TUBE             |         |
| 6,8-10,17-22    | 881077S     | ELR LOCK ROD/ARM ASSEMBLY       |         |
| 25,26           | 881500      | LATCH BLOCK ASSM                | RIGHT   |
| 25,26           | 881501      | LATCH BLOCK ASSM                | LEFT    |
| 1-6,11-16,20-24 | 882413      | ELR PIVOT BLOCK ASSM            | RIGHT   |
| 1-6,11-16,20-24 | 882414      | ELR PIVOT BLOCK ASSM            | LEFT    |
| 1-26            | 881422      | ELR PIVOT ASSM RIGHT            | RIGHT   |
| 1-26            | 881423      | ELR PIVOT ASSM LEFT             | LEFT    |
| 1-30            | 641452S     | ELR HNGR W/O PAD PNTD MNL       | RIGHT   |
| 1-30            | 641453S     | ELR HNGR W/O PAD PNTD MNL       | LEFT    |
| 31A             | 923082S     | ELR HNGR ASSM 14-15 W /PAD (PR) | PAIR    |
| 31B             | 923041S     | ELR HNGR ASSM W/PAD 16-17"      | PAIR    |
| 31C             | 923061S     | ELR HNGR ASSM W/PAD 18-20"      | PAIR    |
| 32A             | 923088S     | ELR HNGR REG EXT 14 15" F/PLT   | PAIR    |
| 32B             | 923084S     | ELR HNGR REG EXT 16 17" F/PLT   | PAIR    |
| 32C             | 923085S     | ELR HNGR REG EXT 18-20" F/PLT   | PAIR    |
| 16,23-26        | 881521      | S/A LATCH KIT CASTING RIGHT     | RIGHT   |
| 16,23-26        | 881522      | S/A LATCH KIT CASTING LEFT      | LEFT    |

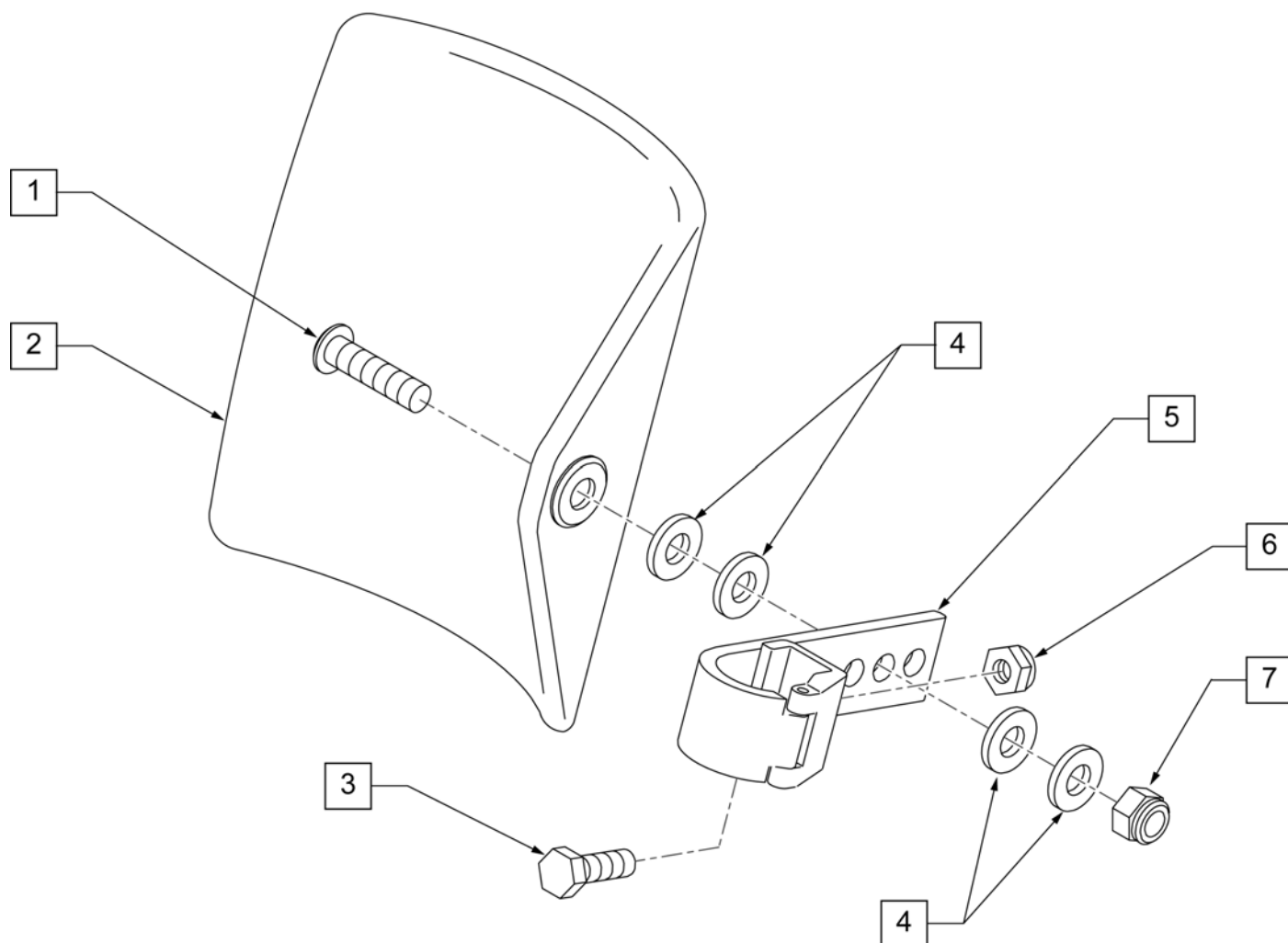


| Pos. | Item Number | Description                      |
|------|-------------|----------------------------------|
| 1    | 106980      | BHCS 5/16-18 X 1-1/4 ROHS        |
| 2    | 755014      | CALF PAD TROUGH STYLE 14"/16"    |
| 3    | 107013      | HHCS 1/4-28 X 5/8 ROHS           |
| 4    | 561016      | 1" WASHER 5/16" HOLE             |
| 5    | 881413      | CALF PAD BRACKET ASSEMBLY        |
| 6    | 106934      | NUT NYLOCK 1/4-28 ROHS           |
| 7    | 202117      | 5/16-18 NYLOCK NUT THICK BLK     |
| 1-7  | 881414      | CALF PAD J-HOOK 14"-15" ASSEMBLY |

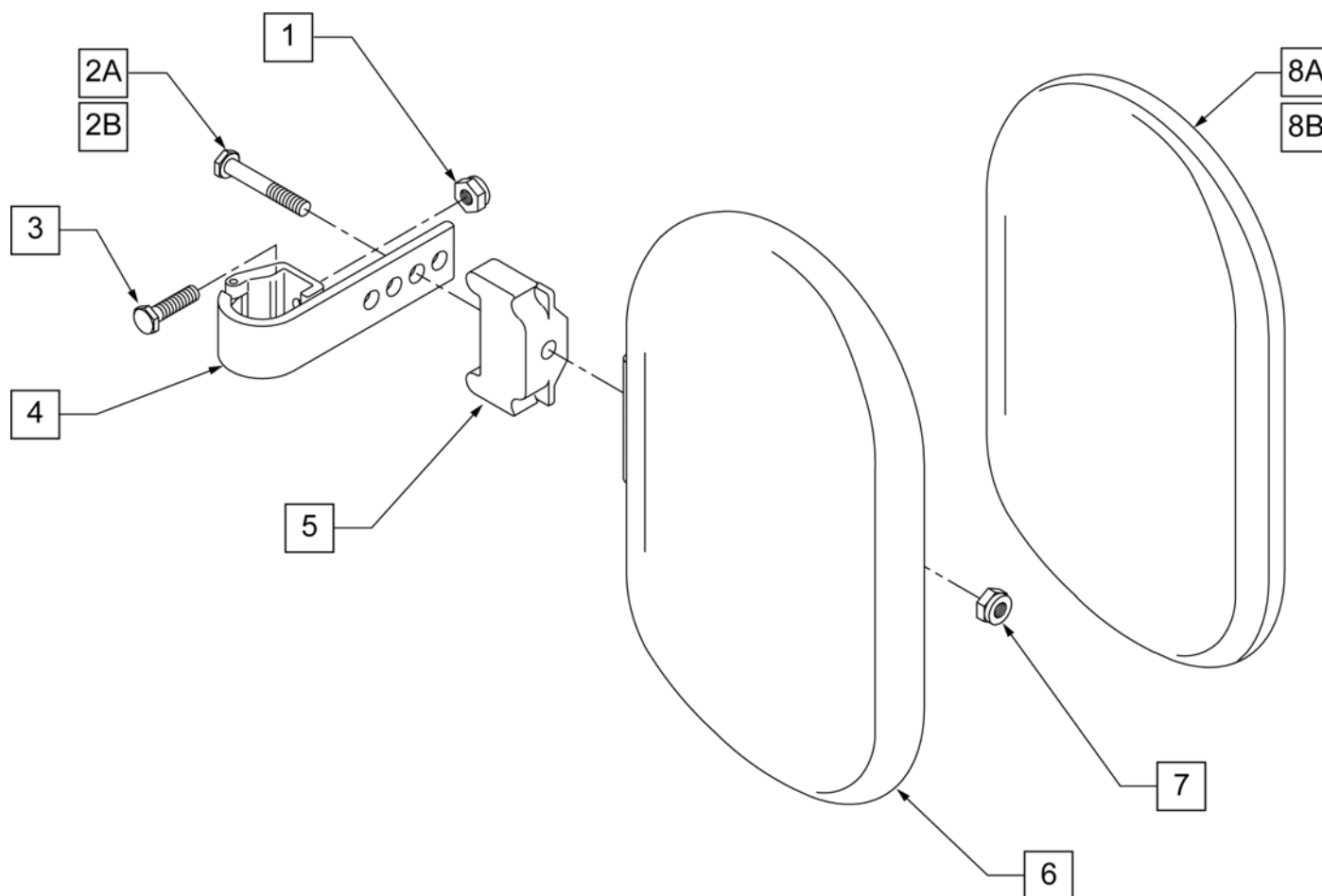

| Pos.         | Item Number | Description                      | Remarks           |
|--------------|-------------|----------------------------------|-------------------|
| 1            | 106934      | NUT NYLOCK 1/4-28 ROHS           |                   |
| 2A           | 107018      | HHCS 5/16-18 X 2-1/2 ROHS        | 16"-17" wide only |
| 2B           | 107019      | HHCS 5/16-18 X 3-1/2 ROHS        | 18"-20" wide only |
| 3            | 107013      | HHCS 1/4-28 X 5/8 ROHS           |                   |
| 4            | 881413      | CALF PAD BRACKET ASSEMBLY        |                   |
| 5            | 560199      | ELR CALF HINGE SPACER            | 18"-20" wide only |
| 6            | 560108      | CALF PAD MOLDED 16"-20"          |                   |
| 7            | 106936      | NUT NYLOCK 5/16-18 ROHS          |                   |
| 8A           | 751045      | CALF PAD COVER                   |                   |
| 8B           | 751046      | CALF PAD COVER (BREEZY)          |                   |
| 1,2A,3,4,6,7 | 881416      | CALF MOLDED 16"-17" ASSEMBLY     |                   |
| 1,2B,3-7     | 881412      | CALF PAD MOLDED 18"-20" ASSEMBLY |                   |

**NOTE:** IF REPLACING AN EXTENSION TUBE ON AN MANUAL ALR PLEASE REFER TO THE ROUND EXT TUBE PARTS MANUAL PAGE.

**NOTE:** WHEN ORDERING REPLACEMENT EXTENSION TUBES PLEASE VERIFY IF YOU HAVE THE OLD STYLE DOUBLE D EXTENSION TUBE OR THE NEW STYLE DOUBLE D EXTENSION TUBE. THIS CAN BE DONE BY REVIEWING THE HANGER THE EXTENSION TUBE IS USED ON. IF THE HANGER UTILIZES A SINGLE SET SCREW AT THE BOTTOM OF THE HANGER TUBE TO ATTACH THE EXTENSION TUBE TO THE HANGER, THEN THIS UTILIZES THE NEW STYLE DOUBLE D EXT TUBE ALL OTHER HANGERS REQUIRING A DOUBLE D EXTENSION TUBE UTILIZE THE OLD STYLE.

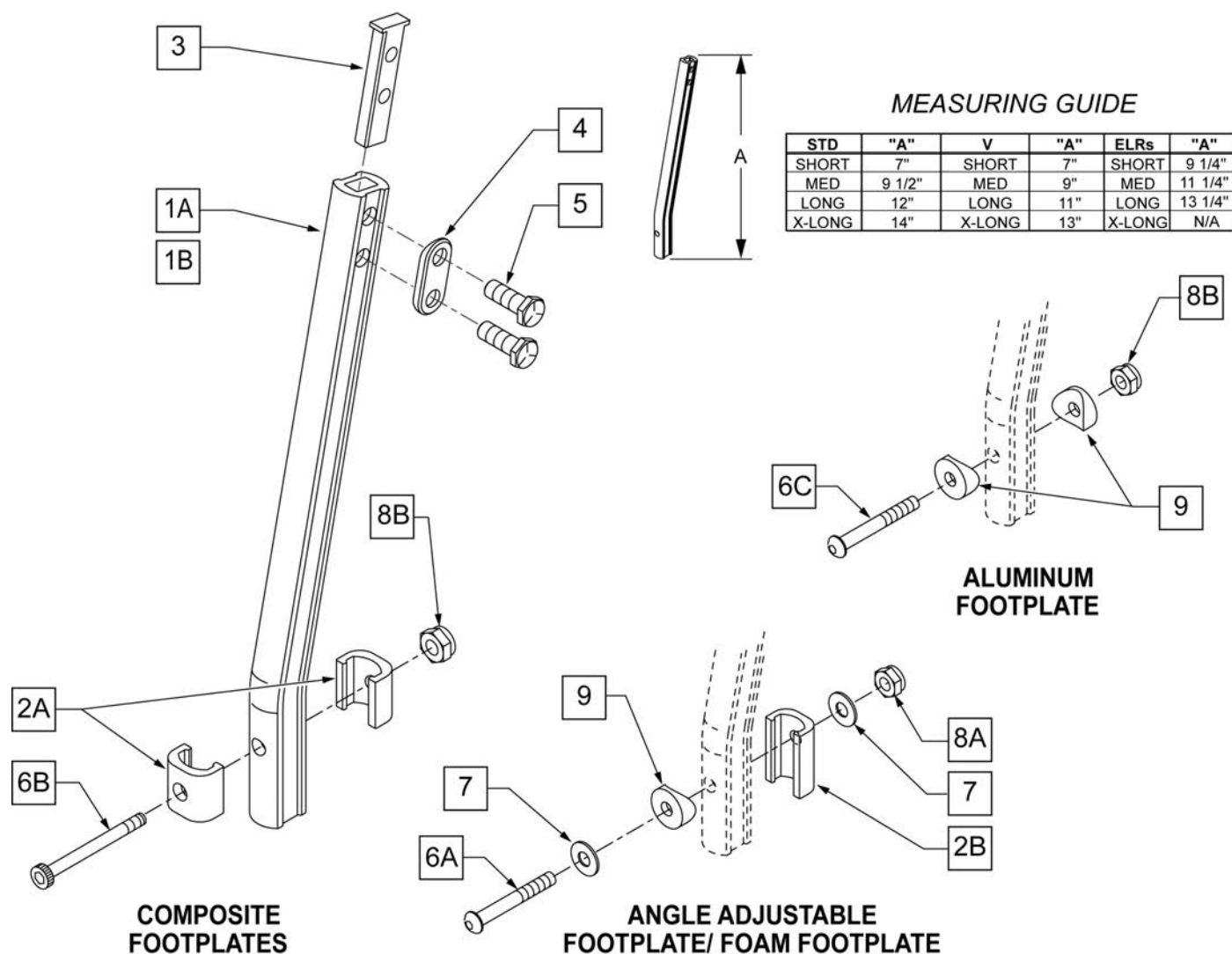


| Pos. | Item Number | Description                 | Remarks                           |
|------|-------------|-----------------------------|-----------------------------------|
| 1A   | 550819      | EXT TUBE S/A ONLY SHT       | Short Ext-Swing Away Hanger       |
| 1A   | 550820      | EXT TUBE S/A ONLY MED       | Med Ext-Swing Away Hanger         |
| 1A   | 550821      | EXT TUBE S/A ONLY LONG      | Long Ext-Swing Away Hanger        |
| 1A   | 550822      | EXT TUBE S/A ONLY X-LONG    | X-Long Ext-Swing Away Hanger      |
| 1B   | 550834      | EXT TUBE V IN ONLY D2 SHT R | Right Short Ext-Swing Away Hanger |
| 1B   | 550835      | EXT TUBE V IN ONLY D2 SHT L | Left Short Ext-Swing Away Hanger  |




| Pos.      | Item Number | Description                          | Remarks                                 |
|-----------|-------------|--------------------------------------|-----------------------------------------|
| 1B        | 550836      | EXT TUBE V IN ONLY D2 MED R          | Right Med Ext-Swing Away Hanger         |
| 1B        | 550837      | EXT TUBE V IN ONLY D2 MED L          | Left Med Ext-Swing Away Hanger          |
| 1B        | 550838      | EXT TUBE V IN ONLY D2 LONG R         | Right Long Ext-Swing Away Hanger        |
| 1B        | 550839      | EXT TUBE V IN ONLY D2 LONG L         | Left Long Ext-Swing Away Hanger         |
| 1B        | 550840      | EXT TUBE V IN ONLY D2 X-LONG R       | Right X-Long Ext-Swing Away Hanger      |
| 1B        | 550841      | EXT TUBE V IN ONLY D2 X-LONG L       | Left X-Long Ext-Swing Away Hanger       |
| 2A        | 483718      | HANGER EXTENSION SADDLE              | USE W/ COMPOSITE FTPLT                  |
| 2B        | 483720      | HANGER EXTENSION SADDLE LONG         | USE W/ ANGLE ADJ FTPLT                  |
| 1C        | 550764      | S/A HANGER EXT D2 SHORT BLACK        | Short Ext ELR Hangers                   |
| 1C        | 550765      | S/A HANGER EXT D2 MED BLACK          | Medium Ext ELR Hangers                  |
| 1C        | 550766      | S/A HANGER EXT D2 LONG BLACK         | Long Ext ELR Hangers                    |
| 3         | 500078      | INSERT D2 EXT TUBE                   |                                         |
| 4         | 483717      | HANGER SADDLE                        |                                         |
| 5         | 221102      | 1/4-28 x 3/4 HEX HEAD C/S BLK W/PTCH |                                         |
| 6A        | 106962      | 1/4-28 x 1 3/4 BH C/S BLK ZN         | USE W/ ANGLE ADJ FTPLT                  |
| 6B        | 226165      | 1/4 -28 x 2 1/2 SOC C/S              | USE W/ COMPOSITE FTPLT                  |
| 6C        | 226147      | 1/4 -28 X 1.85 BH C/S                |                                         |
| 7         | 106945      | 1/4" SAE WASHER                      |                                         |
| 8A        | 106941      | 1/4 -28 AUTOMATION NUT               | USE W/ ANGLE ADJ FTPLT                  |
| 8B        | 106935      | 1/4 -28 NYLOCK NUT THIN              | USE W/ COMPOSITE FTPLT                  |
| 9         | 561055      | 3/4 SADDLE 1" OD                     | USE W/ ANGLE ADJ FTPLT                  |
| 1A,2A-5   | 921759      | EXT TUBE S/A SHT ASSM PCKT           | USE W/ COMPOSITE FTPLT                  |
| 1A,2A-5   | 921760      | EXT TUBE S/A MED ASSM PCKT           | USE W/ COMPOSITE FTPLT                  |
| 1A,2A-5   | 921761      | EXT TUBE S/A LONG ASSM PCKT          | USE W/ COMPOSITE FTPLT                  |
| 1A,2A-5   | 921762      | EXT TUBE S/A X-LONG ASSM PCKT        | USE W/ COMPOSITE FTPLT                  |
| 1B,2A-5   | 921751      | EXT TUBE "V" D2 SHT R ASSM PCKT      | RIGHT - USE W/ COMPOSITE FTPLT V HANGER |
| 1B,2A-5   | 921752      | EXT TUBE "V" D2 SHT L ASSM PCKT      | LEFT - USE W/ COMPOSITE FTPLT V HANGER  |
| 1B,2A-5   | 921753      | EXT TUBE "V" D2 MED R ASSM PCKT      | RIGHT - USE W/ COMPOSITE FTPLT V HANGER |
| 1B,2A-5   | 921754      | EXT TUBE "V" D2 MED L ASSM PCKT      | LEFT - USE W/ COMPOSITE FTPLT V HANGER  |
| 1B,2A-5   | 921755      | EXT TUBE "V" D2 LONG R ASSM PCKT     | RIGHT - USE W/ COMPOSITE FTPLT V HANGER |
| 1B,2A-5   | 921756      | EXT TUBE "V" D2 LONG L ASSM PCKT     | LEFT - USE W/ COMPOSITE FTPLT V HANGER  |
| 1B,2A-5   | 921757      | EXT TUBE "V" D2 X-LONG R ASSM PCKT   | RIGHT - USE W/ COMPOSITE FTPLT V HANGER |
| 1B,2A-5   | 921758      | EXT TUBE "V" D2 X-LONG L ASSM PCKT   | LEFT - USE W/ COMPOSITE FTPLT V HANGER  |
| 1A,2A-7,9 | 921772      | EXT TUBE S/A SHT ANG ADJ PCKT        | USE W/ ANGLE ADJ FTPLT                  |
| 1A,2A-7,9 | 921773      | EXT TUBE S/A MED ANG ADJ PCKT        | USE W/ ANGLE ADJ FTPLT                  |
| 1A,2A-7,9 | 921774      | EXT TUBE S/A LONG ANG ADJ PCKT       | USE W/ ANGLE ADJ FTPLT                  |
| 1A,2A-7,9 | 921775      | EXT TUBE S/A X-LONG ANG ADJ PCKT     | USE W/ ANGLE ADJ FTPLT                  |
| 1B,2A-7,9 | 921764      | EXT TUBE "V" D2 SHT R ANG ADJ PCKT   | RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER |
| 1B,2A-7,9 | 921765      | EXT TUBE "V" D2 SHT L ANG ADJ PCKT   | LEFT - USE W/ ANGLE ADJ FTPLT V HANGER  |
| 1B,2A-7,9 | 921766      | EXT TUBE "V" D2 MED R ANG ADJ PCKT   | RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER |
| 1B,2A-7,9 | 921767      | EXT TUBE "V" D2 MED L ANG ADJ PCKT   | LEFT - USE W/ ANGLE ADJ FTPLT V HANGER  |
| 1B,2A-7,9 | 921768      | EXT TUBE "V" D2 LONG R ANG ADJ PCKT  | RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER |
| 1B,2A-7,9 | 921769      | EXT TUBE "V" D2 LONG L ANG ADJ PCKT  | LEFT - USE W/ ANGLE ADJ FTPLT V HANGER  |
| 1B,2A-7,9 | 921770      | EXT TUBE "V" D2 X-LONG R ANG AD PCKT | RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER |
| 1B,2A-7,9 | 921771      | EXT TUBE "V" D2 X-LONG L ANG AD PCKT | LEFT - USE W/ ANGLE ADJ FTPLT V HANGER  |

**NOTE:** Left Side Shown

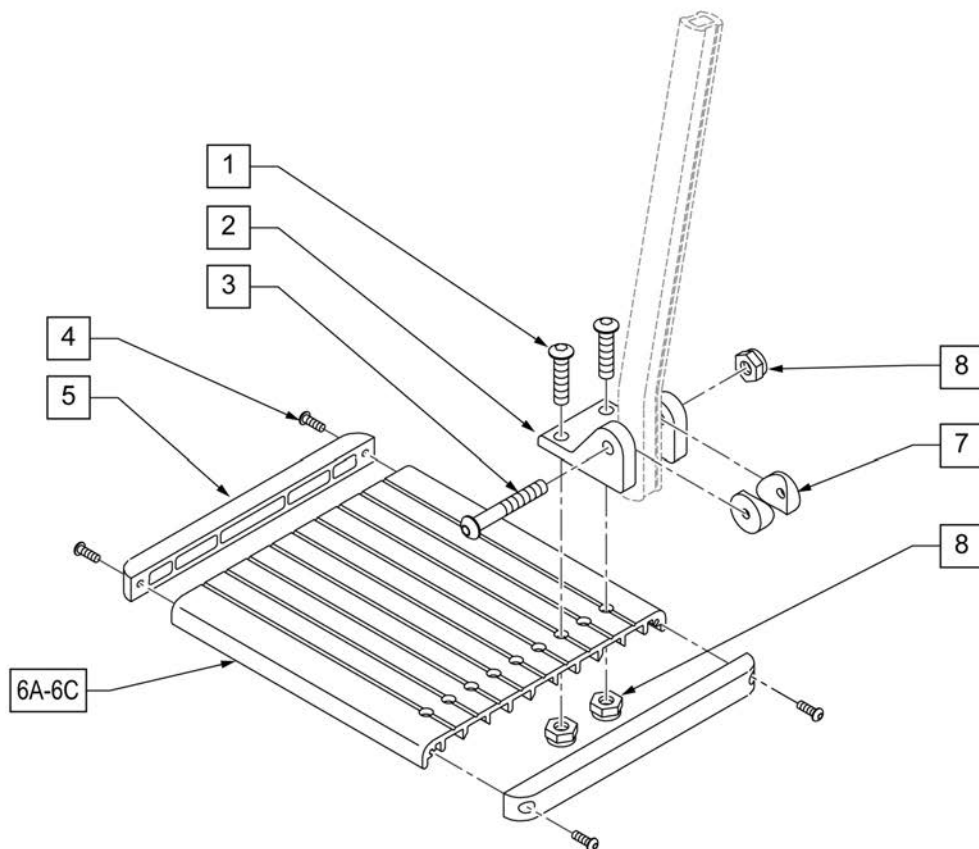

| Pos.     | Item Number | Description                    | Remarks                    |
|----------|-------------|--------------------------------|----------------------------|
| 1        | 106959      | 1/4-28 x 5/8 BH C/S            |                            |
| 2        | 550739      | FOOTPLATE MOUNTING BRACKET     |                            |
| 3        | 226147      | 1/4-28 x 1.85 BH C/S           |                            |
| 4        | 281043      | #4 x 5/8 PAN PHIL S/TAP        |                            |
| 5        | 560101      | END CAPS FOOTREST              |                            |
| 7        | 561003      | 3/4" SADDLE THIN               |                            |
| 8        | 106935      | 1/4-28 NYLOCK NUT THIN         |                            |
| 4,5,6A   | 882337      | ELR FOOTPLATE ASSEMBLY 14 -15" |                            |
| 4,5,6B   | 882338      | ELR FOOTPLATE ASSEMBLY 16 -17" |                            |
| 4,5,6C   | 882339      | ELR FOOTPLATE ASSEMBLY 18 -20" |                            |
| 1-5,6A-8 | 921400S     | ELR FOOTPLATE KIT 14 -15"      | Does not include Extension |
| 1-5,6B-8 | 921401S     | ELR FOOTPLATE KIT 16 -17"      | Does not include Extension |
| 1-5,6C-8 | 921402S     | ELR FOOTPLATE KIT 18 -20"      | Does not include Extension |

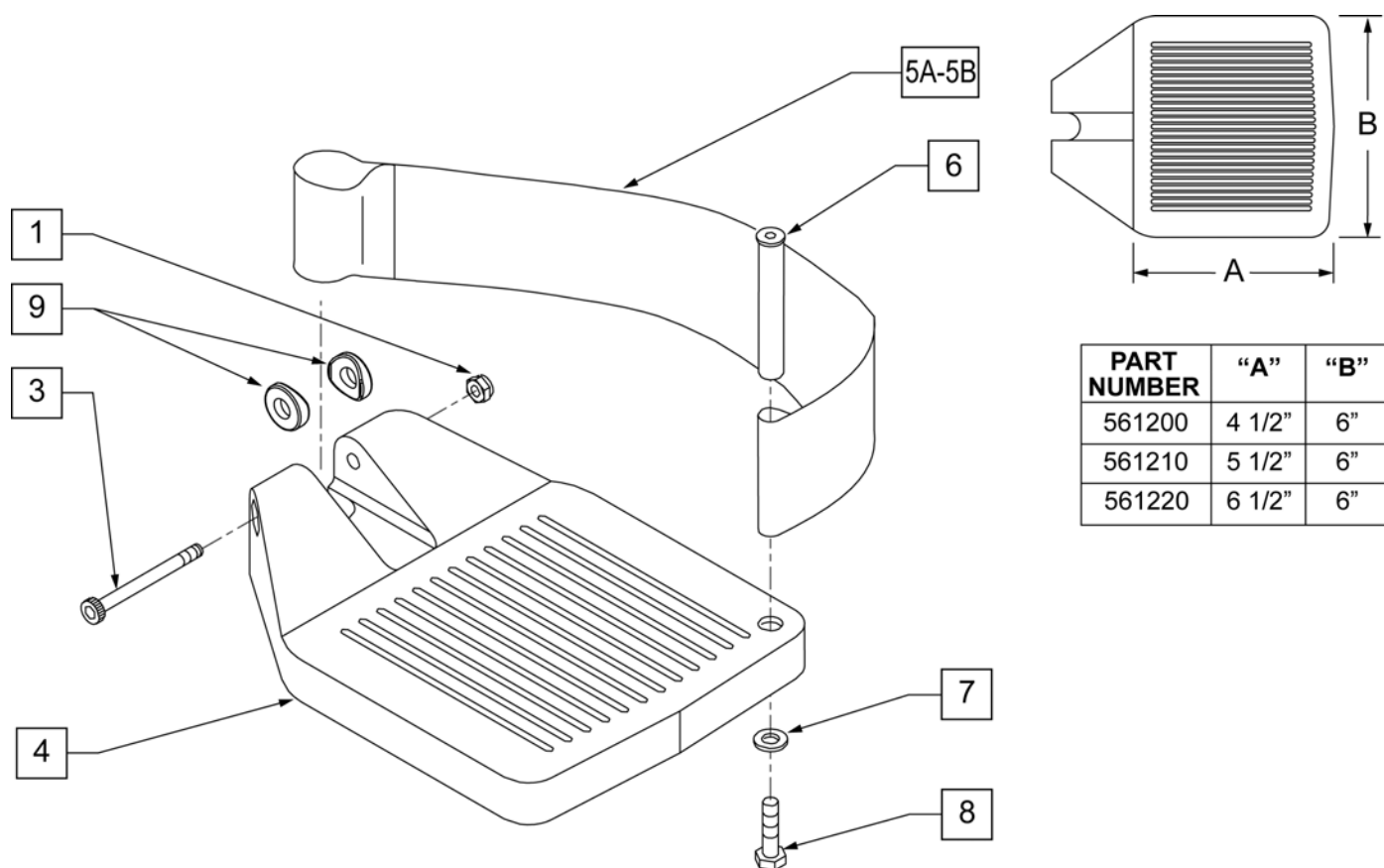

| Pos.  | Item Number | Description                                   | Remarks                                 |
|-------|-------------|-----------------------------------------------|-----------------------------------------|
| 1     | 106935      | 1/4 - 28 NYLOCK NUT                           |                                         |
| 3     | 226165      | 1/4 - 28 x 2 1/2 SOC C/S                      |                                         |
| 4     | 561200      | COMPOSITE FOOTPLATE 14 -15 (16 -18 V )        | 14 -15 OR 16 -18V                       |
| 4     | 561210      | COMPOSITE FOOTPLATE 16 -17 (19 -20 V )        | 16 -17 OR 19 -20V                       |
| 4     | 561220      | COMPOSITE FOOTPLATE 18 -20                    | 18 -20                                  |
| 5A    | 741151      | HEEL LOOP 14-15 COMPOSITE (14 -15 or 16 -18V) | 14 -15 OR 16 -18V                       |
| 5A    | 741156      | HEEL LOOP 16 -17 or 19 -20V                   | 16 -17 OR 19 -20V                       |
| 5A    | 741153      | HEEL LOOP 18-20 COMPOSITE/14-15 ANG ADJ       | 18 -20                                  |
| 5A    | 902270      | HEEL LOOP COMPOSITE FT 14-15                  | PAIR 14 -15 OR 16 -18V                  |
| 5A    | 902271      | HEEL LOOP COMPOSITE FT 16-17                  | PAIR 16 -17 OR 19 -20V                  |
| 5A    | 902272      | HEEL LOOP COMPOSITE FT 18-20                  | PAIR 18 -20                             |
| 5B    | 741154      | HEEL LOOP 14-15 COMPOSITE HEMI                | 14 -15 HEMI                             |
| 5B    | 741155      | HEEL LOOP 16-17 COMPOSITE HEMI                | 16 -17 HEMI                             |
| 5B    | 741153      | HEEL LOOP 18-20 COMPOSITE HEMI                | 18 -10 HEMI                             |
| 5B    | 902260      | HEEL LOOPS HEMI COMPOSITE 14-15               | PAIR 14 -15 HEMI                        |
| 5B    | 902261      | HEEL LOOPS HEMI COMPOSITE 16-17               | PAIR 16 -17 HEMI                        |
| 5B    | 902262      | HEEL LOOPS HEMI COMPOSITE 18-20               | PAIR 18 -20 HEMI                        |
| 6     | 561040      | HEEL LOOP PEG                                 |                                         |
| 7     | 106944      | 1/4 AN WASHER FLAT THIN                       |                                         |
| 8     | 106958      | HHCS 1/4-20 X 1 ROHS                          |                                         |
| 9     | 561003      | 3/4" SADDLE THIN                              | USED W/ ROUND EXT TUBES ONLY            |
| 1,3,4 | 921180      | COMPOSITE FOOTPLATE ASSM 14-15                | RIGHT (14 -15 OR 16 -18V) W/O HEEL LOOP |
| 1,3,4 | 921181      | COMPOSITE FOOTPLATE ASSM 14-15                | LEFT (14 -15 OR 16 -18V) W/O HEEL LOOP  |
| 1,3,4 | 921182      | COMPOSITE FOOTPLATE ASSM 16-17                | RIGHT (16 -17 OR 19 -20V) W/O HEEL LOOP |
| 1,3,4 | 921183      | COMPOSITE FOOTPLATE ASSM 16-17                | LEFT (16 -17 OR 19 -20V) W/O HEEL LOOP  |

| Pos.     | Item Number | Description                                | Remarks                                |
|----------|-------------|--------------------------------------------|----------------------------------------|
| 1,3,4    | 921184      | COMPOSITE FOOTPLATE ASSM 18-20             | RIGHT (18 -20) W/O HEEL LOOP           |
| 1,3,4    | 921185      | COMPOSITE FOOTPLATE ASSM 18-20             | LEFT (18 -20) W/O HEEL LOOP            |
| 1,3,4    | 921151      | COMPOSITE FOOTPLATE ASSM 14-15/16-18V      | PAIR (14 -15 OR 16 -18V) W/O HEEL LOOP |
| 1,3,4    | 921152      | COMPOSITE FOOTPLATE ASSM 16-17/19-20V      | PAIR (16 -17 OR 19 -20V) W/O HEEL LOOP |
| 1,3,4    | 921153      | COMPOSITE FOOTPLATE ASSM 18-20             | PAIR (18 -20) W/O HEEL LOOP            |
| 1,3,4A-8 | 921190      | COMPOSITE FOOTPLATE ASSM HL LP 14-15       | RIGHT (14 -15 OR 16 -18V) W/ HEEL LOOP |
| 1,3,4A-8 | 921191      | COMPOSITE FOOTPLATE ASSM HL LP 14-15       | LEFT (14 -15 OR 16 -18V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921192      | COMPOSITE FOOTPLATE ASSM HL LP 16-17       | RIGHT (16 -17 OR 19 -20V) W/ HEEL LOOP |
| 1,3,4A-8 | 921193      | COMPOSITE FOOTPLATE ASSM HL LP 16-17       | LEFT (16 -17 OR 19 -20V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921194      | COMPOSITE FOOTPLATE ASSM HL LP 18-20       | RIGHT (18 -20) W/ HEEL LOOP            |
| 1,3,4A-8 | 921195      | COMPOSITE FOOTPLATE ASSM HL LP 18-20       | LEFT (18 -20) W/ HEEL LOOP             |
| 1,3,4A-8 | 921161      | COMPOSITE FOOTPLATE ASSM HL 14-15/16-18V   | PAIR (14 -15 OR 16 -18V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921162      | COMPOSITE FOOTPLATE ASSM HL 16-17/19-20V   | PAIR (16 -17 OR 19 -20V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921163      | COMPOSITE FOOTPLATE ASSM HL 18-20          | PAIR (18 -20) W/ HEEL LOOP             |
| 1,3,4B-8 | 921140      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15  | RIGHT (14 -15) W/ HEEL LOOP            |
| 1,3,4B-8 | 921141      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15  | LEFT (14 -15) W/ HEEL LOOP             |
| 1,3,4B-8 | 921142      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17  | RIGHT (16 -17) W/ HEEL LOOP            |
| 1,3,4B-8 | 921143      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17  | LEFT (16 -17) W/ HEEL LOOP             |
| 1,3,4B-8 | 921144      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20  | RIGHT (18 -20) W/ HEEL LOOP            |
| 1,3,4B-8 | 921145      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20  | LEFT (18 -20) W/ HEEL LOOP             |
| 1,3,4B-8 | 921167      | COMPOSITE FOOTPLATE ASSM HEMI HL LPS 14-15 | PAIR (14 -15) W/ HEEL LOOP             |
| 1,3,4B-8 | 921168      | COMPOSITE FOOTPLATE ASSM HEMI HL LPS 16-17 | PAIR (16 -17) W/ HEEL LOOP             |
| 1,3,4B-8 | 921169      | COMPOSITE FOOTPLATE ASSM HEMI HL LPS 18-20 | PAIR (18 -20) W/ HEEL LOOP             |


**NOTE:** FOR HIGH-MOUNT ADULT ANGLE-ADJUSTABLE FOOTPLATE, SEE "HIGH-MOUNT EXTENSION TUBE" PAGE.

**NOTE:** ORDER ITEM 4B FOR CHAIRS PRIOR TO 8/13/01.

**NOTE:** ONLY 1 WASHER PRIOR TO 8/13/01.

**NOTE:** ORDER PART 226167 PRIOR TO 8/13/01.

**NOTE:** See attached REFERENCE GUIDE.

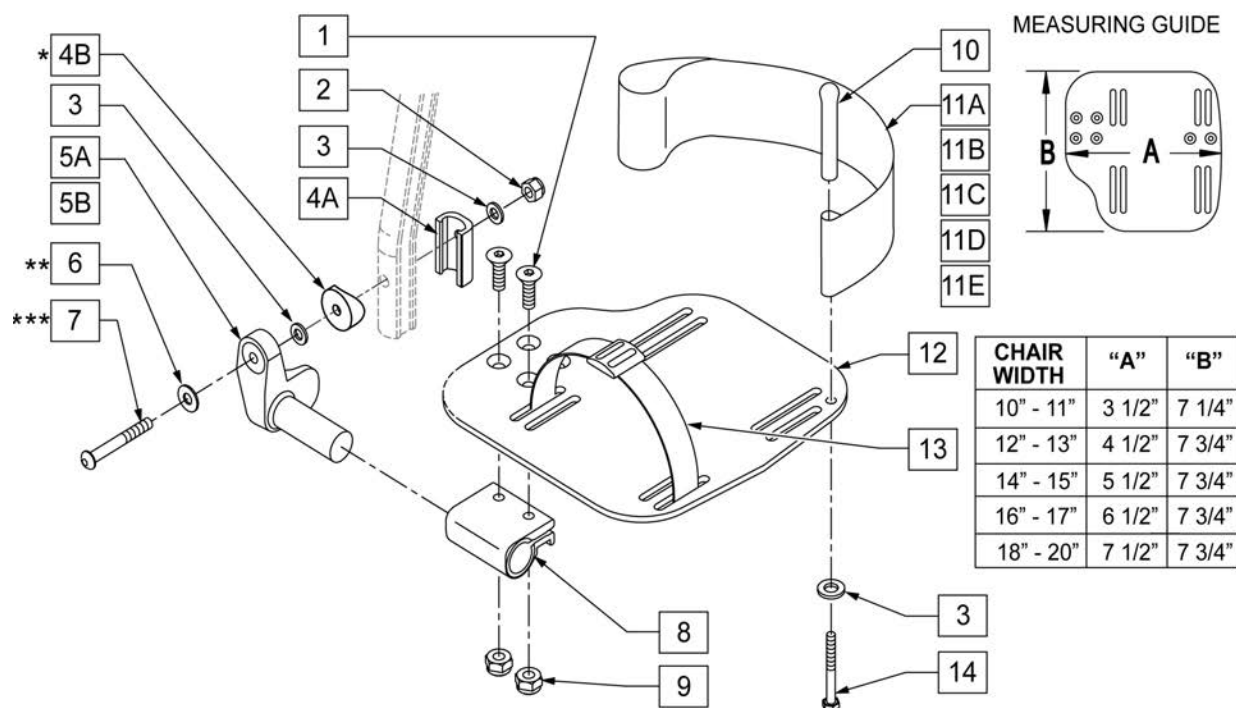


| Pos. | Item Number | Description           | Remarks |
|------|-------------|-----------------------|---------|
| 1    | 106978      | 1/4-28 x 7/8 FH C/S   |         |
| 2    | 106941      | 1/4-28 AUTOMATION NUT |         |

| Pos.                    | Item Number | Description                         | Remarks                                    |
|-------------------------|-------------|-------------------------------------|--------------------------------------------|
| 3                       | 106943      | WASHER FLAT AN 1/4 ROHS             |                                            |
| 4A                      | 483720      | HANGER EXTENSION SADDLE LONG        | USE W/ DOUBLE D EXT TUBE                   |
| 4B                      | 561055      | 3/4" SADDLE 1" OD                   | USE W/ ROUND EXT TUBE                      |
| 5                       | 541002      | FTRST MOUNTING BRKT-3/8 RAD R       | RIGHT                                      |
| 5                       | 541003      | FTRST MOUNTING BRKT-3/8 RAD L       | LEFT                                       |
| 6                       | 106945      | WASHER FLAT SAE 1/4 ROHS            |                                            |
| 7A                      | 106962      | BHCS 1/4-28 X 1-3/4 ROHS            | USE W/ DOUBLE D EXT TUBE                   |
| 7B                      | 226167      | 1/4-28 x 1.625 BH C/S BLK           | USE W/ ROUND EXT TUBE                      |
| 8                       | 550350      | 3/4" CLMP ADJ FT PLT                |                                            |
| 9                       | 106935      | NUT NYLOCK THIN 1/4-28 ROHS         |                                            |
| 10                      | 561040      | HEEL LOOP PEG                       |                                            |
| 11A                     | 741154      | HEEL LOOP 14-15 CMPST HEMI          | USED W/ 10-11" ADULT ANG ADJ FTPLTS        |
| 11B                     | 741155      | HEEL LOOP 16-17 CMPST HEMI          | USED W/ 12 -13" ADULT ANG ADJ FTPLTS       |
| 11C                     | 741153      | H/L 18-20 CMPST/14-15 ANG ADJ       | USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS |
| 11D                     | 741253      | HEEL LOOP ADLT ADJ F/PLT 18-20      | USED W/ 18 -20" ADULT ANG ADJ FTPLTS       |
| 12                      | 500656      | ADLT ANGL ADJ F/PLT 10/11" R        | RIGHT                                      |
| 12                      | 500657      | ADLT ANGL ADJ F/PLT 10/11" L        | LEFT                                       |
| 12                      | 500658      | ADLT ANGL ADJ F/PLT 12/13" R        | RIGHT                                      |
| 12                      | 500659      | ADLT ANGL ADJ F/PLT 12/13" L        | LEFT                                       |
| 12                      | 500660      | ADLT ANGL ADJ F/PLT 14/15           | RIGHT                                      |
| 12                      | 500661      | ADLT ANGL ADJ F/PLT 14/15           | LEFT                                       |
| 12                      | 500662      | ADLT ANGL ADJ F/PLT 16/17           | RIGHT                                      |
| 12                      | 500663      | ADLT ANGL ADJ F/PLT 16/17           | LEFT                                       |
| 12                      | 500664      | ADLT ANGL ADJ F/PLT 18-20           | RIGHT                                      |
| 12                      | 500665      | ADLT ANGL ADJ F/PLT 18-20           | LEFT                                       |
| 13                      | 740260      | TOE/ANKLE STRAP ADJ ADLT F/PLT      |                                            |
| 14                      | 106958      | 1/4-20 x 1 HEX C/S                  |                                            |
| 1-3,5,6,8,9,12          | 820526      | AD ANG ADJ F/PLT R 12"/13" OMIT HL  | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820536      | AD ANG ADJ F/PLT L 12"/13" OMT HL   | LEFT                                       |
| 1-3,5,6,8,9,12          | 820460      | FP ADLT ADJ 14/15 OMIT HL           | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820462      | FP ADLT ADJ 14/15 OMIT HL           | LEFT                                       |
| 1-3,5,6,8,9,12          | 820464      | FP ADLT ADJ 16/17 OMIT HL           | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820466      | FP ADLT ADJ 16/17 OMIT HL           | LEFT                                       |
| 1-3,5,6,8,9,12          | 820468      | FP ADLT ADJ 18-20 OMIT HL           | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820470      | FP ADLT ADJ 18-20 OMIT HL           | LEFT                                       |
| 1-3,5,6,8,9,12          | 921466      | ADLT ANGL ADJ F/PLT 12"/13" OMIT HL | PAIR                                       |
| 1-3,5,6,8,9,12          | 921320      | FP ADLT ADJ 14/15 OMIT HL           | PAIR                                       |
| 1-3,5,6,8,9,12          | 921322      | FP ADLT ADJ 16/17 OMIT HL           | PAIR                                       |
| 1-3,5,6,8,9,12          | 921324      | FP ADLT ADJ 18-20 OMIT HL           | PAIR                                       |
| 1-3,5,6,8,9,12          | 820522      | AD ANGL ADJ F/PLT R 12/13" H/L      | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820532      | AD ANGL ADJ F/PLT L 12/13" H/L      | LEFT                                       |
| 1-3,5,6,8-10,11C, 12,14 | 820461      | FP ADLT ADJ 14/15 HL                | RIGHT                                      |
| 1-3,5,6,8-10,11C, 12,14 | 820463      | FP ADLT ADJ 14/15 HL                | LEFT                                       |
| 1-3,5,6,8-10,11C, 12,14 | 820465      | FP ADLT ADJ 16/17 HL                | RIGHT                                      |
| 1-3,5,6,8-10,11C, 12,14 | 820467      | FP ADLT ADJ 16/17 HL                | LEFT                                       |
| 1-3,5,6,8-10,11D, 12,14 | 820469      | FP ADLT ADJ 18-20 HL                | RIGHT                                      |
| 1-3,5,6,8-10,11D, 12,14 | 820471      | FP ADLT ADJ 18-20 HL                | LEFT                                       |
| 1-3,5,6,8-10,11B, 12,14 | 921462      | ADLT ANGL ADJ F/PLATE 12"/13" HL    | PAIR                                       |
| 1-3,5,6,8-10,11C, 12,14 | 921321      | FP ADLT ADJ 14/15 HL                | PAIR                                       |
| 1-3,5,6,8-10,11C, 12,14 | 921323      | FP ADLT ADJ 16/17 HL                | PAIR                                       |
| 1-3,5,6,8-10,11D, 12,14 | 921325      | FP ADLT ADJ 18-20 HL                | PAIR                                       |
| 1-3,6,7A,9-10,14        | 101533      | ADLT ANG ADJ FP D2 HDWR KIT         | FOR DOUBLE D EXT TUBES                     |
| 1-3B,4B,6,7B,9,10,14    | 101534      | ADLT ANG ADJ FP RND HDWR KIT        | FOR ROUND EXT TUBES (Q2 HP ONLY)           |

**NOTE:** \* ORDER ITEM 8 PRIOR TO 8/13/01.

\*\* ORDER ITEM 5 PART 226167 PRIOR TO 8/13/01.

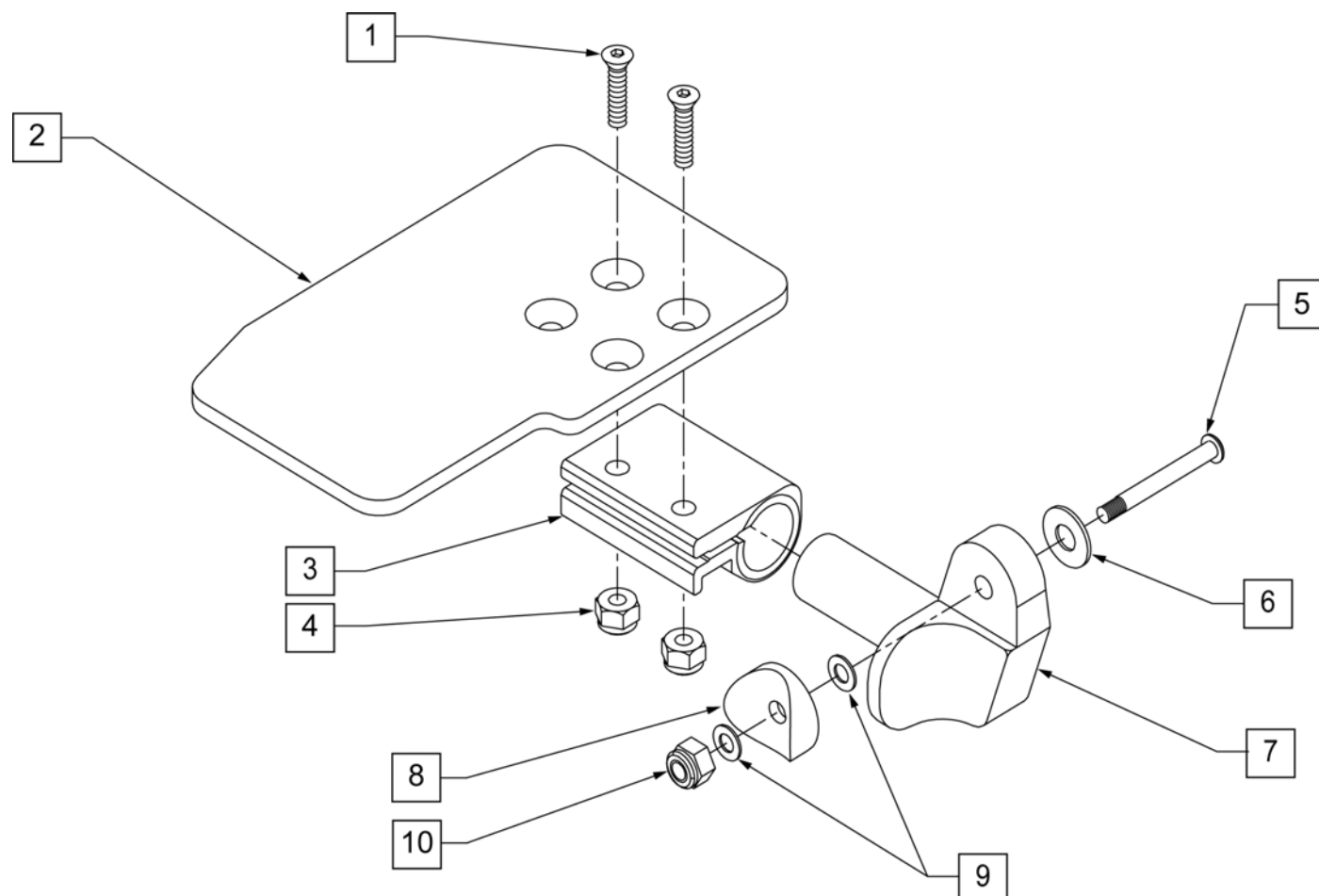

| Pos. | Item Number | Description                    | Remarks |
|------|-------------|--------------------------------|---------|
| 1    | 106978      | 1/4-28 X 7/8 FH C/S            |         |
| 2    | 500635      | KIDS ANGLE ADJ FOOTPLATE 10" R |         |
| 2    | 500636      | KIDS ANGLE ADJ FOOTPLATE 10" L |         |
| 2    | 500637      | KIDS ANGLE ADJ FOOTPLATE 11" R |         |
| 2    | 500638      | KIDS ANGLE ADJ FOOTPLATE 11" L |         |
| 2    | 500639      | KIDS ANGLE ADJ FOOTPLATE 12" R |         |
| 2    | 500640      | KIDS ANGLE ADJ FOOTPLATE 12" L |         |
| 2    | 500641      | KIDS ANGLE ADJ FOOTPLATE 13" R |         |
| 2    | 500642      | KIDS ANGLE ADJ FOOTPLATE 13" L |         |
| 2    | 500643      | KIDS ANGLE ADJ FOOTPLATE 14" R |         |
| 2    | 500644      | KIDS ANGLE ADJ FOOTPLATE 14" L |         |
| 2    | 500645      | KIDS ANGLE ADJ FOOTPLATE 15" R |         |
| 2    | 500646      | KIDS ANGLE ADJ FOOTPLATE 15" L |         |
| 2    | 500647      | KIDS ANGLE ADJ FOOTPLATE 16" R |         |
| 2    | 500648      | KIDS ANGLE ADJ FOOTPLATE 16" L |         |
| 3    | 550350      | 3/4" CLAMP ADJ FOOTPLATE       |         |
| 4    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS    |         |


| Pos.     | Item Number | Description                             | Remarks |
|----------|-------------|-----------------------------------------|---------|
| 5**      | 106962      | 1/4-28 X 1.85 BH C/S BLK ZN             |         |
| 6        | 106945      | 1/4 SAE WASHER                          |         |
| 7        | 541002      | FOOTREST MOUNTING BRACKET - 3/8" RADIUS | RIGHT   |
| 7        | 541003      | FOOTREST MOUNTING BRACKET - 3/8" RADIUS | LEFT    |
| 8*       | 561055      | 3/4" SADDLE 1" OD                       |         |
| 9        | 106944      | 1/4 AN WASHER FLAT THIN                 |         |
| 10       | 106941      | 1/4-28 AUTOMATION NUT                   |         |
| 1-7,9,10 | 828850      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 10" R  | RIGHT   |
| 1-7,9,10 | 828851      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 10" L  | LEFT    |
| 1-7,9,10 | 828852      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 11" R  | RIGHT   |
| 1-7,9,10 | 828853      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 11" L  | LEFT    |
| 1-7,9,10 | 828854      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 12" R  | RIGHT   |
| 1-7,9,10 | 828855      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 12" L  | LEFT    |
| 1-7,9,10 | 828856      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 13" R  | RIGHT   |
| 1-7,9,10 | 828857      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 13" L  | LEFT    |
| 1-7,9,10 | 828858      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 14" R  | RIGHT   |
| 1-7,9,10 | 828859      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 14" L  | LEFT    |
| 1-7,9,10 | 828860      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 15" R  | RIGHT   |
| 1-7,9,10 | 828861      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 15" L  | LEFT    |
| 1-7,9,10 | 828862      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 16" R  | RIGHT   |
| 1-7,9,10 | 828863      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 16" L  | LEFT    |
| 1-7,9,10 | 921280      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 10"    | PAIR    |
| 1-7,9,10 | 921281      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 11"    | PAIR    |
| 1-7,9,10 | 921282      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 12"    | PAIR    |
| 1-7,9,10 | 921283      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 13"    | PAIR    |
| 1-7,9,10 | 921284      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 14"    | PAIR    |
| 1-7,9,10 | 921285      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 15"    | PAIR    |
| 1-7,9,10 | 921286      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 16"    | PAIR    |



# 90 DEG ELEVATING HANGER


[11/2009]

NOTE: Order Item 27 (561002 ) when NOT using Footrest Extension.

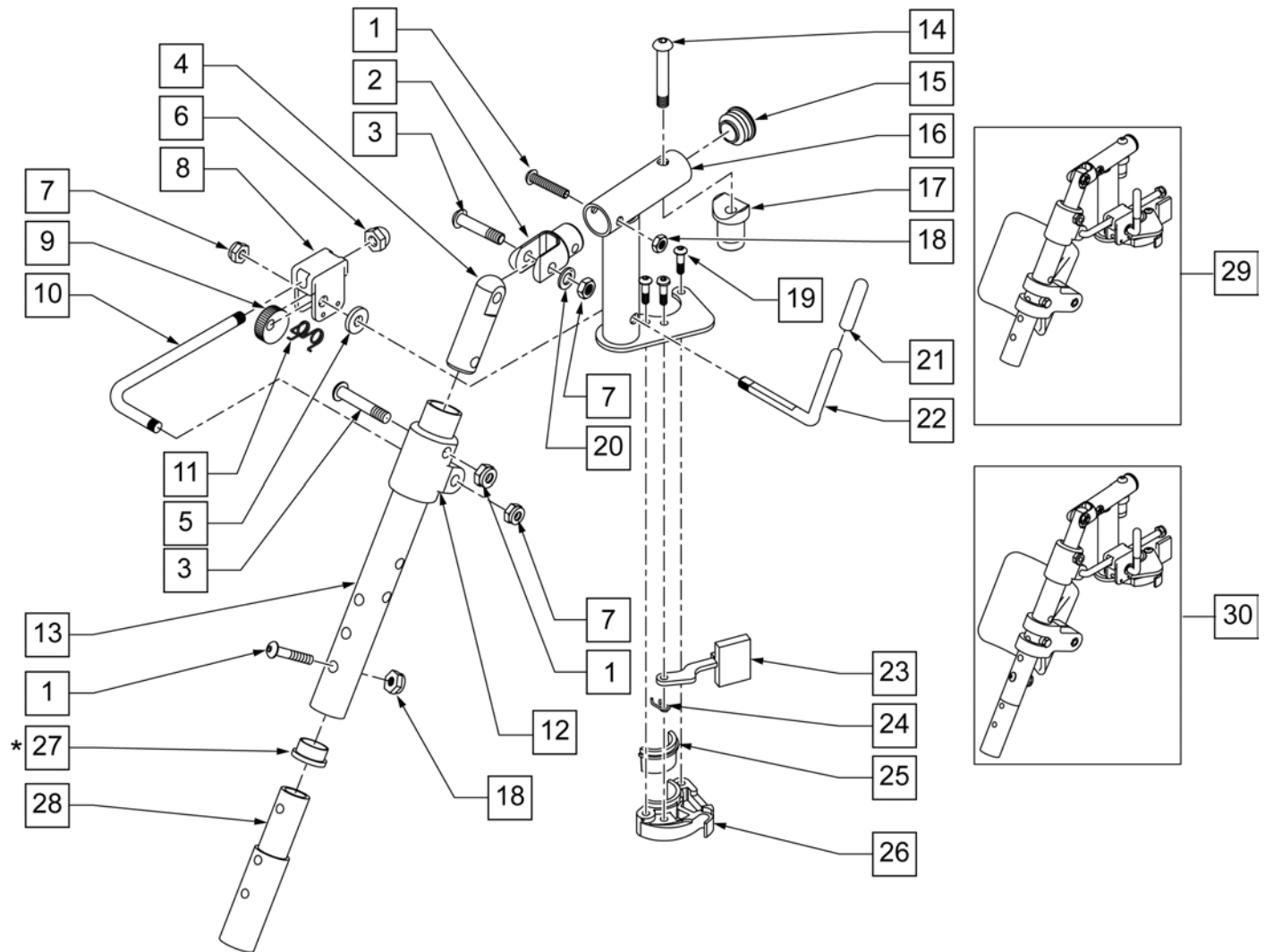


| Pos. | Item Number | Description                    | Remarks       |
|------|-------------|--------------------------------|---------------|
| 1    | 107027      | BHCS 1/4-28 X 1-1/4 SHLDR ROHS |               |
| 2    | 505281      | ELR CLEVIS                     |               |
| 3    | 107037      | BHCS 5/16-24 X 1-1/2 SHLD ROHS |               |
| 4    | 540252      | ELR PIVOT PRS                  |               |
| 5    | 561059      | 3/4 OD x .318 ID WASHER        |               |
| 6    | 202109      | 5/16-24 ACORN NUT HEX          |               |
| 7    | 106939      | NUT NYLOCK THIN 5/16-24 ROHS   |               |
| 8    | 880990      | ELR LOCK HOUSING W/PIN         |               |
| 9    | 481119      | ELR CAM LOCKING POWDERED METAL |               |
| 10   | 481505      | ELR LOCK ROD 90D               |               |
| 11   | 321106      | ELR TORSION SPRING             |               |
| 12   | 560430      | ELR ROD UPPER PIVOT            |               |
| 13   | 505097S     | ELR KIDS TUBE                  | SPECIFY COLOR |
| 14   | 106983      | BHCS 5/16-18 X 2 SHLDR ROHS    |               |

| Pos.              | Item Number | Description                    | Remarks |
|-------------------|-------------|--------------------------------|---------|
| 15                | 561062      | 1" INSERT PLUG .065 RIBS       |         |
| 17                | 561197      | PLASTIC PIVOT SADDLE           |         |
| 18                | 106935      | NUT NYLOCK THIN 1/4-28 ROHS    |         |
| 19                | 107035      | BHCS 10-24 X .725 PL ROHS      |         |
| 20                | 263100      | 5/16 AN WASHER FLAT            |         |
| 21                | 561049      | ELR RELEASE ROD COVER          |         |
| 23                | 561079      | S/A LATCH LEVER                |         |
| 24                | 321136      | S/A SPRING L RED               |         |
| 24                | 321137      | S/A SPRING R CLEAR             |         |
| 27                | 561002      | 1" INSERT PLUG .083            |         |
| 29                | 923067S     | ELR HNGR ASSM 90D W/PAD        | PAIR    |
| 30                | 923068S     | ELR HNGR ASSM 90D W/PAD 2" EXT | PAIR    |
| 1,18,28           | 111505      | 2" FTREST EXT W/ HDWR          |         |
| 25,26             | 881500      | LATCH BLOCK ASSM R             | RIGHT   |
| 25,26             | 881501      | LATCH BLOCK ASSM L             | LEFT    |
| 21,22             | 882411      | ELR 90D ACTUATION ARM ASSM     |         |
| 1-4,7,14-20,23-28 | 882413      | ELR PIVOT BLOCK ASSM R         | RIGHT   |
| 1-4,7,14-20,23-28 | 882414      | ELR PIVOT BLOCK ASSM L         | LEFT    |
| 1-26              | 641458S     | ELR HNGR 90D W/O PAD PNTD R    | RIGHT   |
| 1-26              | 641459S     | ELR HNGR 90D W/O PAD PNTD L    | LEFT    |
| 1-26,28           | 641468S     | HANGER ELR 90 W/O PAD R 2" EXT | RIGHT   |
| 1-26,28           | 641469S     | HANGER ELR 90 W/O PAD L 2" EXT | LEFT    |
| 19,23-26          | 881521      | S/A LATCH KIT CASTING RIGHT    | RIGHT   |
| 19,23-26          | 881522      | S/A LATCH KIT CASTING LEFT     | LEFT    |

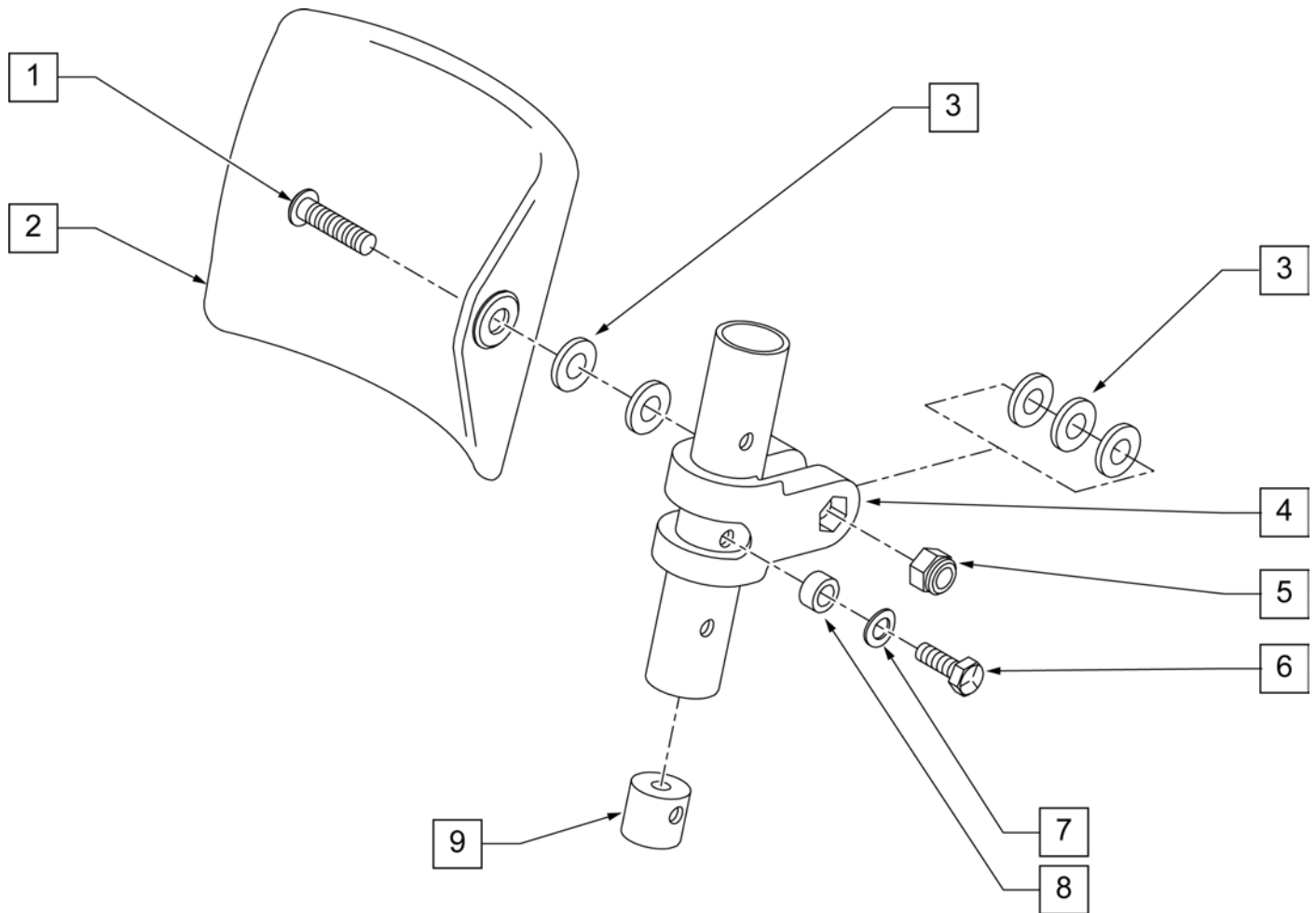


| Pos. | Item Number | Description                 |
|------|-------------|-----------------------------|
| 1    | 226196      | 5/16-18 x 2 BH C/S ZN       |
| 2    | 755013      | CALF PAD THROUGH STYLE KIDS |
| 3    | 561016      | 1" WASHER 5/16" HOLE        |
| 4    | 561097      | ELR ELEVATING TUBE CLAMP    |
| 5    | 106937      | 5/16 -18 NYLOCK NUT THIN    |
| 6    | 107017      | 1/4-20 x 1 1/4 G8 HEX C/S   |
| 7    | 262130      | 6mm WASHER WAVE             |
| 8    | 483027      | 1/4 x 1/4 ALUMINUM SPACER   |
| 9    | 500151      | KIDS ALR RETAINING PLUG     |
| 1-9  | 880533      | ELR TROUGH PAD ASSM KIDS    |

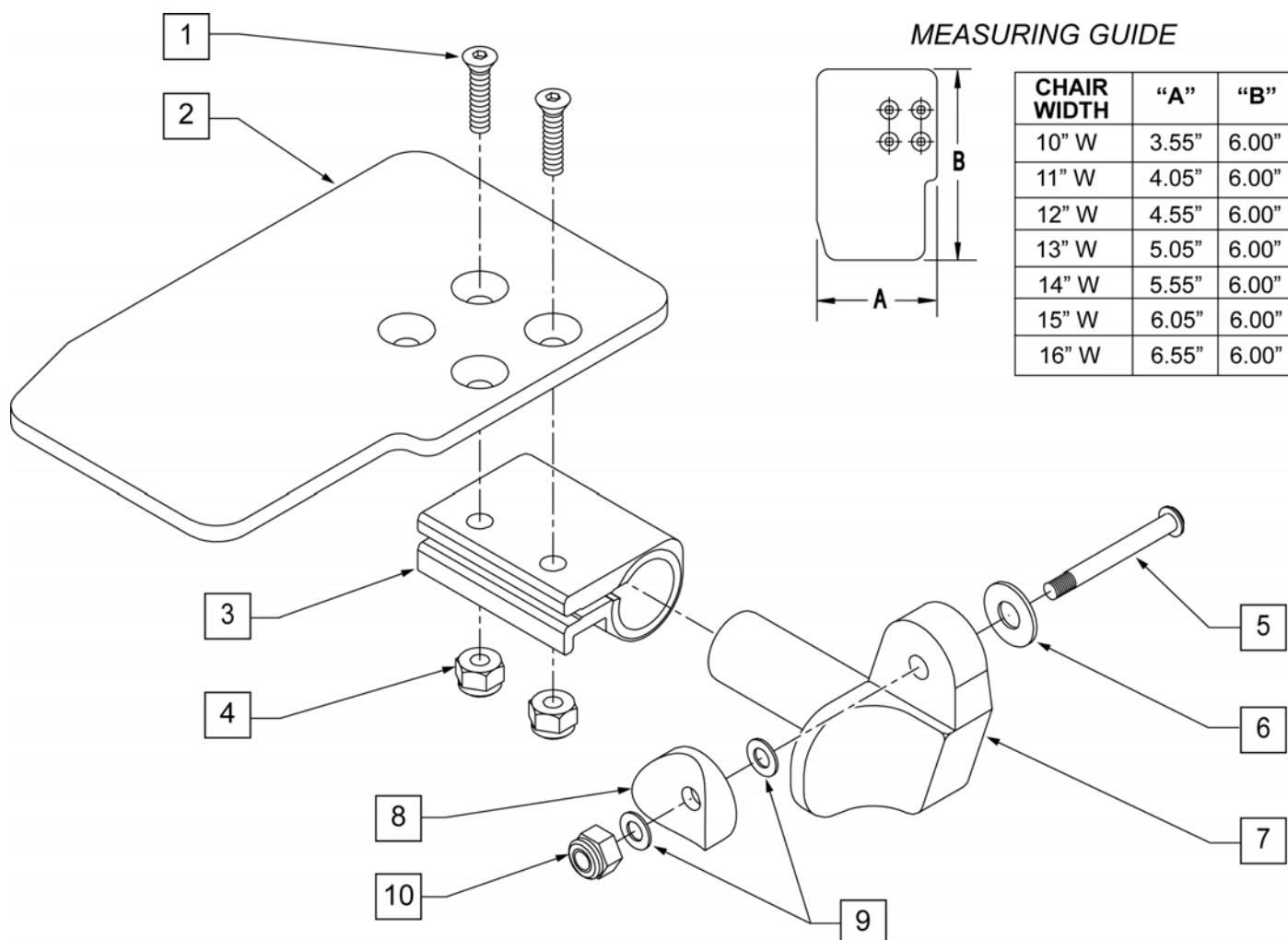

| Pos. | Item Number | Description                 | Remarks | Pos. | Item Number | Description                | Remarks |
|------|-------------|-----------------------------|---------|------|-------------|----------------------------|---------|
| 1    | 106978      | FHCS 1/4-28 X 7/8 ROHS      |         | 7    | 541001      | FTRST MOUNTING BRKT 1 RAD  | LEFT    |
| 2    | 500635      | KIDS ANGL ADJ F/PLT 10      | RIGHT   | 8    | 561004      | 1 SADDLE 1/4 HOLE          |         |
| 2    | 500636      | KIDS ANGL ADJ F/PLT 10      | LEFT    | 9    | 106943      | WASHER FLAT THIN AN 1/4    |         |
| 2    | 500637      | KIDS ANGL ADJ F/PLT 11      | RIGHT   | 10   | 106941      | NUT AUTOMATION 1/4-28 ROHS |         |
| 2    | 500638      | KIDS ANGL ADJ F/PLT 11      | LEFT    | 1-10 | 828821      | KIDS ANG ADJ F/PLT ASSM 10 | RIGHT   |
| 2    | 500639      | KIDS ANGL ADJ F/PLT 12      | RIGHT   | 1-10 | 828822      | KIDS ANG ADJ F/PLT ASSM 10 | LEFT    |
| 2    | 500640      | KIDS ANGL ADJ F/PLT 12      | LEFT    | 1-10 | 828823      | KIDS ANG ADJ F/PLT ASSM 11 | RIGHT   |
| 2    | 500641      | KIDS ANGL ADJ F/PLT 13      | RIGHT   | 1-10 | 828824      | KIDS ANG ADJ F/PLT ASSM 11 | LEFT    |
| 2    | 500642      | KIDS ANGL ADJ F/PLT 13      | LEFT    | 1-10 | 828825      | KIDS ANG ADJ F/PLT ASSM 12 | RIGHT   |
| 2    | 500643      | KIDS ANGL ADJ F/PLT 14      | RIGHT   | 1-10 | 828826      | KIDS ANG ADJ F/PLT ASSM 12 | LEFT    |
| 2    | 500644      | KIDS ANGL ADJ F/PLT 14      | LEFT    | 1-10 | 828827      | KIDS ANG ADJ F/PLT ASSM 13 | RIGHT   |
| 2    | 500645      | KIDS ANGL ADJ F/PLT 15      | RIGHT   | 1-10 | 828828      | KIDS ANG ADJ F/PLT ASSM 13 | LEFT    |
| 2    | 500646      | KIDS ANGL ADJ F/PLT 15      | LEFT    | 1-10 | 828829      | KIDS ANG ADJ F/PLT ASSM 14 | RIGHT   |
| 2    | 500647      | KIDS ANGL ADJ F/PLT 16      | RIGHT   | 1-10 | 828830      | KIDS ANG ADJ F/PLT ASSM 14 | LEFT    |
| 2    | 500648      | KIDS ANGL ADJ F/PLT 16      | LEFT    | 1-10 | 828831      | KIDS ANG ADJ F/PLT ASSM 15 | RIGHT   |
| 3    | 550350      | 3/4 CLAMP ADJ FOOTPLATE     |         | 1-10 | 828832      | KIDS ANG ADJ F/PLT ASSM 15 | LEFT    |
| 4    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS |         | 1-10 | 828833      | KIDS ANG ADJ F/PLT ASSM 16 | RIGHT   |
| 5    | 226147      | 1/4-28 X 1.85 BH C/S BLK ZN |         | 1-10 | 828834      | KIDS ANG ADJ F/PLT ASSM 16 | LEFT    |
| 6    | 106945      | WASHER FLAT SAE 1/4 ROHS    |         | 1-10 | 921226      | KIDS ANG ADJ F/PLT ASSM 10 | PAIR    |
| 7    | 541000      | FTRST MOUNTING BRKT 1 RAD   | RIGHT   | 1-10 | 921227      | KIDS ANG ADJ F/PLT ASSM 11 | PAIR    |


| Pos. | Item Number | Description                | Remarks | Pos. | Item Number | Description                | Remarks |
|------|-------------|----------------------------|---------|------|-------------|----------------------------|---------|
| 1-10 | 921228      | KIDS ANG ADJ F/PLT ASSM 12 | PAIR    | 1-10 | 921231      | KIDS ANG ADJ F/PLT ASSM 15 | PAIR    |
| 1-10 | 921229      | KIDS ANG ADJ F/PLT ASSM 13 | PAIR    | 1-10 | 921232      | KIDS ANG ADJ F/PLT ASSM 16 | PAIR    |
| 1-10 | 921230      | KIDS ANG ADJ F/PLT ASSM 14 | PAIR    |      |             |                            |         |

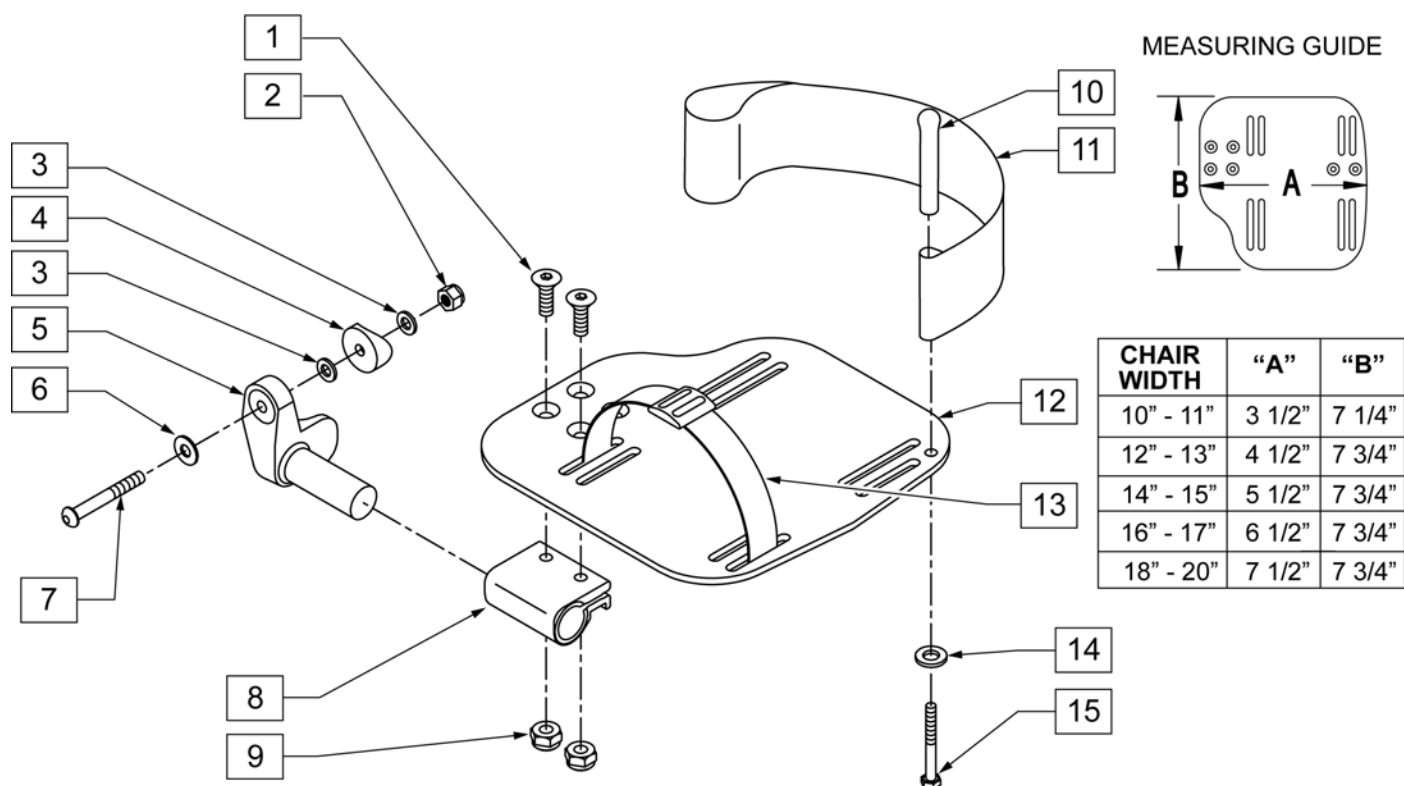

| Pos. | Item Number | Description                         | Remarks                                    |
|------|-------------|-------------------------------------|--------------------------------------------|
| 1    | 106978      | FHCS 1/4-28 X 7/8 ROHS              |                                            |
| 2    | 106941      | NUT AUTOMATION 1/4-28 ROHS          |                                            |
| 3    | 106943      | WASHER FLAT AN 1/4 ROHS             |                                            |
| 4    | 561004      | 1" SADDLE 1/4" HOLE                 |                                            |
| 5    | 541000      | FOOTREST MOUNTING BRKT-1" RAD       | RIGHT                                      |
| 5    | 541001      | FOOTREST MOUNTING BRKT-1" RAD       | LEFT                                       |
| 6    | 106945      | WASHER FLAT SAE 1/4 ROHS            |                                            |
| 7    | 106964      | BHCS 1/4-28 X 2                     |                                            |
| 8    | 550350      | 3/4" CLAMP ADJ FOOTPLATE            |                                            |
| 9    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS         |                                            |
| 10   | 561040      | HEEL LOOP PEG                       |                                            |
| 11   | 741153      | HEEL LOOP 18-20 CMPST/14-15 ANG ADJ | USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS |
| 11   | 741253      | HEEL LOOP ADLT ADJ F/PLT 18-20      | USED W/ 18 -20" ADULT ANG ADJ FTPLTS       |
| 12   | 500656      | ADULT ANGL ADJ F/PLT 10/11          | RIGHT                                      |
| 12   | 500657      | ADULT ANGL ADJ F/PLT 10/11          | LEFT                                       |
| 12   | 500658      | ADULT ANGL ADJ F/PLT 12/13          | RIGHT                                      |
| 12   | 500659      | ADULT ANGL ADJ F/PLT 12/13          | LEFT                                       |
| 12   | 500660      | ADULT ANGL ADJ F/PLT 14/15          | RIGHT                                      |
| 12   | 500661      | ADULT ANGL ADJ F/PLT 14/15          | LEFT                                       |
| 12   | 500662      | ADULT ANGL ADJ F/PLT 16/17          | RIGHT                                      |
| 12   | 500663      | ADULT ANGL ADJ F/PLT 16/17          | LEFT                                       |
| 12   | 500664      | ADULT ANGL ADJ F/PLT 18-20          | RIGHT                                      |
| 12   | 500665      | ADULT ANGL ADJ F/PLT 18-20          | LEFT                                       |
| 13   | 740260      | TOE-ANKLE STRAP ADJ ADULT FOOTPLATE |                                            |
| 14   | 106943      | WASHER FLAT AN 1/4 ROHS             |                                            |

| Pos.       | Item Number | Description                       | Remarks |
|------------|-------------|-----------------------------------|---------|
| 15         | 106958      | HHCS 1/4-20 X 1 ROHS              |         |
| 1-9,12     | 820544      | FP ADULT ADJ 90D 10/11 R          | RIGHT   |
| 1-9,12     | 820554      | FP ADULT ADJ 90D 10/11 L          | LEFT    |
| 1-9,12     | 820546      | FP ADULT ADJ 90D 12/13 R          | RIGHT   |
| 1-9,12     | 820556      | FP ADULT ADJ 90D 12/13 L          | LEFT    |
| 1-9,12     | 820472      | FP ADULT ADJ 90D 14/15 R          | RIGHT   |
| 1-9,12     | 820473      | FP ADULT ADJ 90D 14/15 L          | LEFT    |
| 1-9,12     | 820474      | FP ADULT ADJ 90D 16/17 R          | RIGHT   |
| 1-9,12     | 820475      | FP ADULT ADJ 90D 16/17 L          | LEFT    |
| 1-9,12     | 820476      | FP ADULT ADJ 90D 18-20 R          | RIGHT   |
| 1-9,12     | 820477      | FP ADULT ADJ 90D 18-20 L          | LEFT    |
| 1-9,12     | 921474      | ADULT ANGL ADJ 90D 10/11 OMIT H/L | PAIR    |
| 1-9,12     | 921476      | ADULT ANGL ADJ 90D 12/13 OMIT H/L | PAIR    |
| 1-9,12     | 921330      | ADULT ANGL ADJ 90D 14/15 OMIT H/L | PAIR    |
| 1-9,12     | 921331      | ADULT ANGL ADJ 90D 16/17 OMIT H/L | PAIR    |
| 1-9,12     | 921332      | ADULT ANGL ADJ 90D 18-20 OMIT H/L | PAIR    |
| 1-12,14,15 | 820540      | F/PLT ADULT ADJ 90D 10/11 R W/ HL | RIGHT   |
| 1-12,14,15 | 820550      | F/PLT ADULT ADJ 90D 10/11 L W/ HL | LEFT    |
| 1-12,14,15 | 820542      | F/PLT ADULT ADJ 90D 12/13 R W/ HL | RIGHT   |
| 1-12,14,15 | 820552      | F/PLT ADULT ADJ 90D 12/13 L W/ HL | LEFT    |
| 1-12,14,15 | 820478      | F/PLT ADULT ADJ 90D 14/15 R W/ HL | RIGHT   |
| 1-12,14,15 | 820479      | F/PLT ADULT ADJ 90D 14/15 L W/ HL | LEFT    |
| 1-12,14,15 | 820480      | F/PLT ADULT ADJ 90D 16/17 R W/ HL | RIGHT   |
| 1-12,14,15 | 820481      | F/PLT ADULT ADJ 90D 16/17 L W/ HL | LEFT    |
| 1-12,14,15 | 820482      | F/PLT ADULT ADJ 90D 18-20 R W/ HL | RIGHT   |
| 1-12,14,15 | 820483      | F/PLT ADULT ADJ 90D 18-20 L W/ HL | LEFT    |
| 1-12,14,15 | 921470      | F/PLT ADULT ADJ 90D 10/11 W/ HL   | PAIR    |
| 1-12,14,15 | 921472      | F/PLT ADULT ADJ 90D 12/13 W/ HL   | PAIR    |
| 1-12,14,15 | 921333      | F/PLT ADULT ADJ 90D 14/15 W/ HL   | PAIR    |
| 1-12,14,15 | 921334      | F/PLT ADULT ADJ 90D 16/17 W/ HL   | PAIR    |
| 1-12,14,15 | 921335      | F/PLT ADULT ADJ 90D 18-20 W/ HL   | PAIR    |

**NOTE:** WHEN USING AN ARTICULATING LEGREST ASSEMBLY, THE PULL-TO-LOCK WHEEL LOCK MUST BE USED.

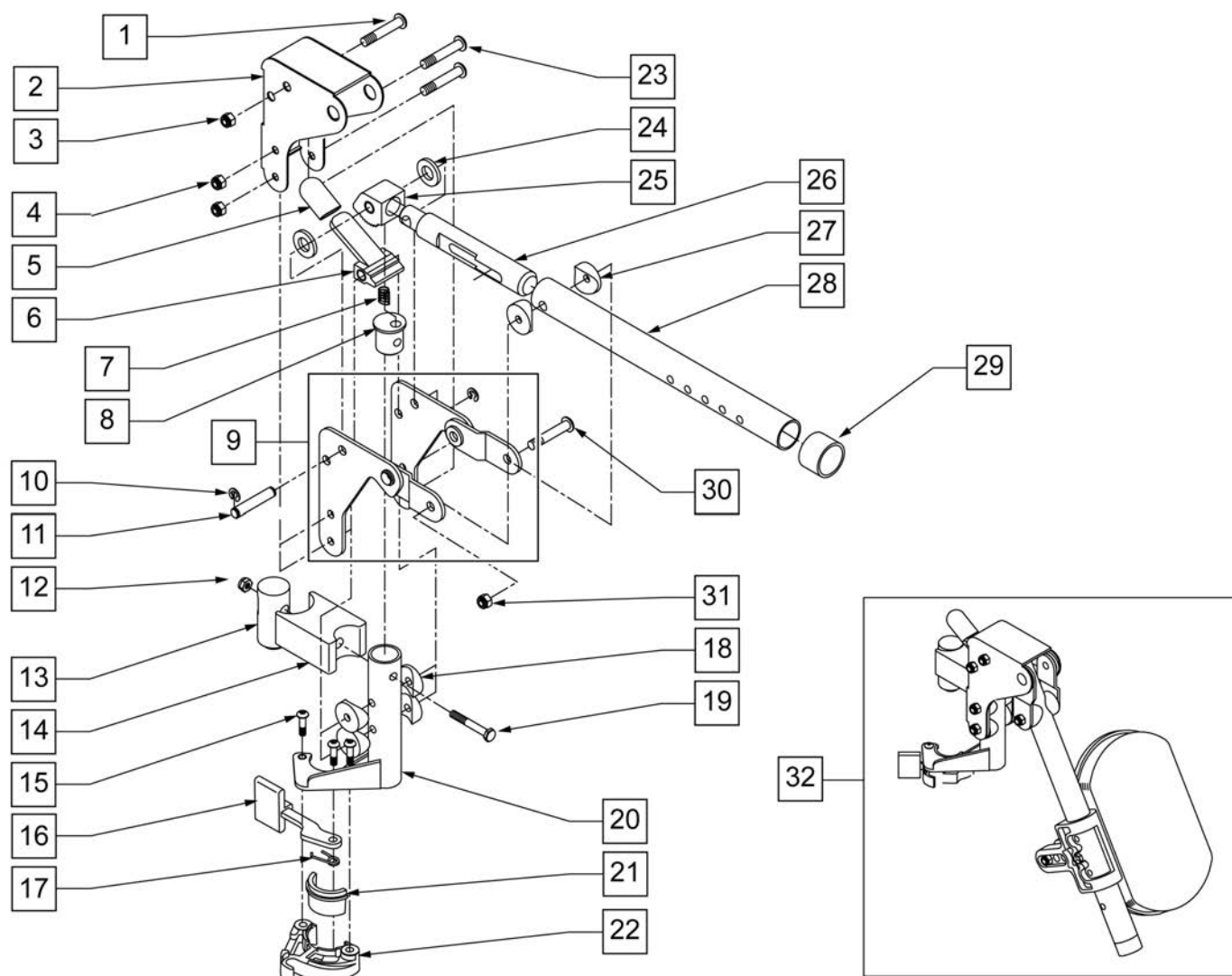


| Pos. | Item Number | Description                   | Remarks |
|------|-------------|-------------------------------|---------|
| 1    | 106983      | 5/16-18 x 2 BH C/S SHT THREAD |         |
| 2    | 562720      | ALR COVER STD                 |         |
| 3    | 106937      | 5/16-18 NYLOCK NUT THIN       |         |
| 4    | 106935      | 1/4-28 NYLOCK NUT THIN        |         |
| 5    | 560501      | ALR RELEASE LEVER COVER       |         |
| 6    | 505902      | ALR RELEASE LEVER             |         |
| 7    | 321141      | 650L x 6 COIL SPRING          |         |
| 8    | 483060      | ALR SPRING PLUG               |         |
| 9    | 880526      | ALR BRACKET W/LINK R          | RIGHT   |
| 9    | 880527      | ALR BRACKET W/LINK L          | LEFT    |
| 10   | 342000      | 1/4\" E-CLIP                  |         |
| 11   | 481052      | ALR RETAINING STUD            |         |
| 12   | 106934      | 1/4-28 NYLOCK NUT             |         |
| 13   | 122991      | ALR PIVOT SADDLE WITH LEDGE   |         |




| Pos.        | Item Number | Description                    | Remarks |
|-------------|-------------|--------------------------------|---------|
| 14          | 122992      | ALR PIVOT SADDLE BLOCK W/NOTCH |         |
| 15          | 226057      | 10/24 x .725 BTTN HD SOC BZ    |         |
| 16          | 561079      | S/A LATCH LEVER                |         |
| 17          | 321136      | S/A SPRING L RED               |         |
| 17          | 321137      | S/A SPRING R CLEAR             |         |
| 18          | 561004      | 1" SADDLE 1/4" HOLE            |         |
| 19          | 107016      | 1/4-28 x 3 G8 HEX C/S          |         |
| 20          | 882466      | ALR S/A TUBE ASSM L            | LEFT    |
| 20          | 882467      | ALR S/A TUBE ASSM R            | RIGHT   |
| 23          | 106964      | 1/4-28 x 2 BH C/S              |         |
| 24          | 561016      | 1" WASHER 5/16" HOLE           |         |
| 25          | 505900      | ALR RATCHET BLOCK              |         |
| 26          | 483062      | ALR RATCHET ROD                |         |
| 27          | 561006      | 1" SADDLE 5/16" HOLE           |         |
| 28          | 505026S     | ALR TUBE ONLY                  |         |
| 29          | 561082      | 1" TUBE END BUSHING THIN       |         |
| 30          | 226195      | 5/16-18 x 1.82 BH C/S          |         |
| 31          | 106937      | NUT NYLOCK THIN 5/16-18 ROHS   |         |
| 21,22       | 881500      | LATCH BLOCK ASSM R             | RIGHT   |
| 21,22       | 881501      | LATCH BLOCK ASSM L             | LEFT    |
| 1-26        | 880092      | HANGER ALR SUB ASSM R          | RIGHT   |
| 1-26        | 880093      | HANGER ALR SUB ASSM L          | LEFT    |
| 1-31        | 641464S     | HANGER ALR W/O PAD PNTD R      | RIGHT   |
| 1-31        | 641465S     | HANGER ALR W/O PAD PNTD L      | LEFT    |
| 32          | 923051S     | ALR HANGER ASSM W/PAD ALUM LB  | PAIR    |
| 15-17,21,22 | 881521      | S/A LATCH KIT CASTING RIGHT    |         |
| 15-17,21,22 | 881522      | S/A LATCH KIT CASTING LEFT     |         |

**NOTE:** WHEN USING AN ARTICULATING LEGREST ASSEMBLY, THE PULL-TO-LOCK WHEEL LOCK MUST BE USED.

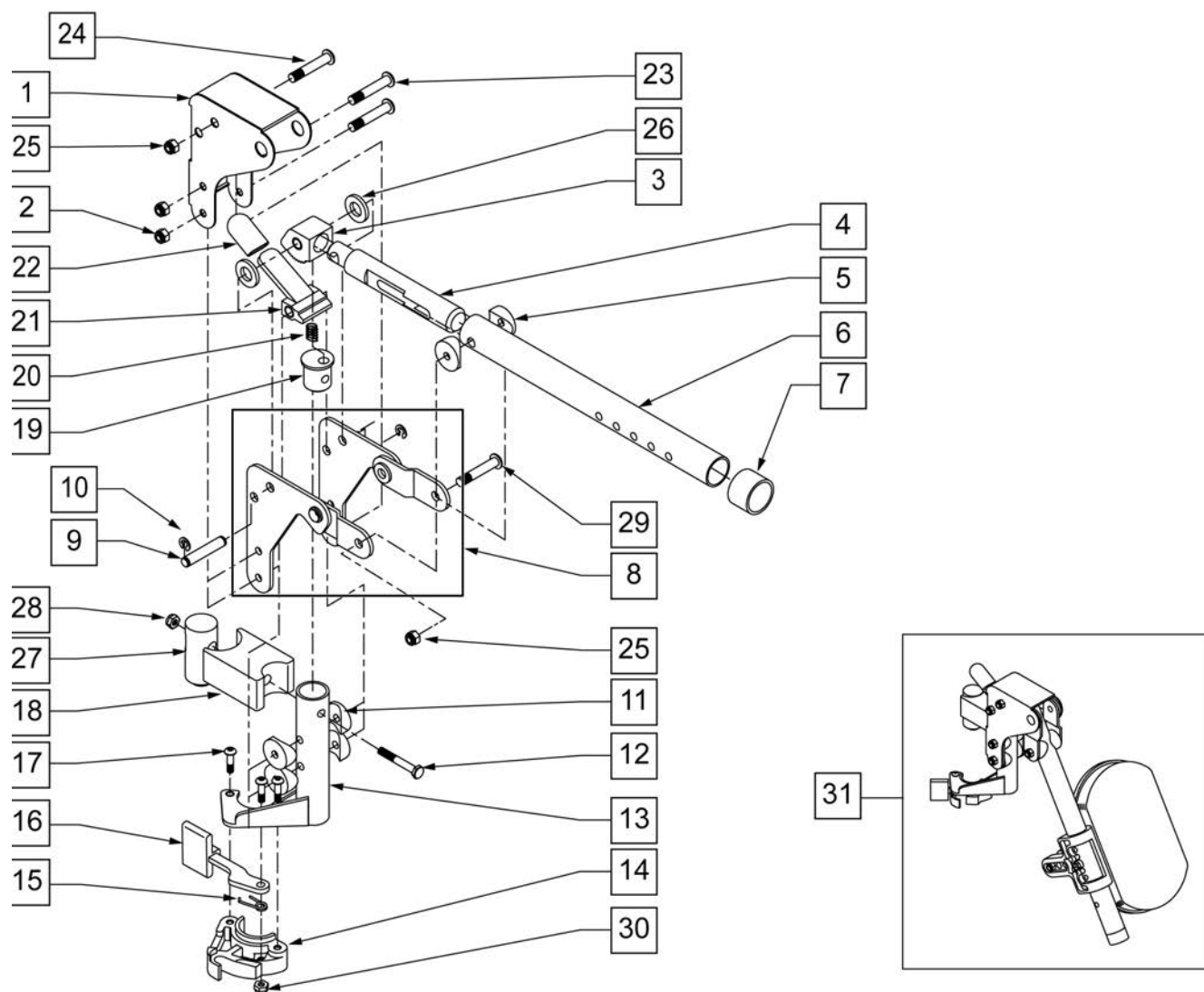

| Pos. | Item Number | Description                   | Remarks |
|------|-------------|-------------------------------|---------|
| 1    | 562720      | ALR COVER STD                 |         |
| 2    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS   |         |
| 3    | 505900      | ALR RATCHET BLOCK             |         |
| 4    | 483062      | ALR RATCHET ROD               |         |
| 5    | 561006      | 1" SADDLE 5/16" HOLE          |         |
| 6    | 505026S     | ALR TUBE ONLY (SPECIFY COLOR) |         |
| 7    | 561082      | 1" TUBE END BUSHING THIN      |         |
| 8    | 880526      | ALR BRACKET W/LINK            | RIGHT   |
| 8    | 880527      | ALR BRACKET W/LINK            | LEFT    |
| 9    | 481052      | ALR RETAINING STUD            |         |
| 10   | 342000      | 1/4" E-CLIP                   |         |


| Pos. | Item Number | Description                           | Remarks |
|------|-------------|---------------------------------------|---------|
| 11   | 561004      | 1" SADDLE 1/4" HOLE                   |         |
| 12   | 107016      | 1/4-28 x 3 G8 HEX C/S                 |         |
| 13   | 882466      | ALR S/A TUBE ASSM L                   | RIGHT   |
| 13   | 882467      | ALR S/A TUBE ASSM R                   | LEFT    |
| 14   | 561076      | S/A LATCH BLOCK                       | RIGHT   |
| 14   | 561078      | S/A LATCH BLOCK                       | LEFT    |
| 15   | 321136      | S/A SPRING L RED                      | LEFT    |
| 15   | 321137      | S/A SPRING R CLEAR                    | RIGHT   |
| 16   | 561079      | S/A LATCH LEVER                       |         |
| 17   | 226057      | 10/24 x .725 BTTN HD SOC BZ           |         |
| 18   | 500178      | ALR PIVOT SADDLE BLOCK                |         |
| 19   | 483060      | ALR SPRING PLUG                       |         |
| 20   | 321141      | .650L x 6 COIL SPRING                 |         |
| 21   | 505902      | ALR RELEASE LEVER                     |         |
| 22   | 560501      | ALR RELEASE LEVER COVER               |         |
| 23   | 106964      | 1/4-28 x 2 BH C/S                     |         |
| 24   | 226217      | 5/16-18 x 2" BH C/S BLK               |         |
| 25   | 106937      | 5/16-18 NYLOCK NUT THIN               |         |
| 26   | 561016      | 1" WASHER 5/16" HOLE                  |         |
| 27   | 500177      | ALR PIVOT SADDLE .125                 |         |
| 28   | 106934      | 1/4-28 NYLOCK NUT                     |         |
| 29   | 226195      | 5/16-18 x 1.82 BH C/S                 |         |
| 30   | 200107      | 10-24 NYLOCK NUT THIN                 |         |
| 1-30 | 641464S     | HANGER ALR W/O PAD PNTD SPECIFY COLOR | RIGHT   |
| 1-30 | 641465S     | HANGER ALR W/O PAD PNTD SPECIFY COLOR | LEFT    |
| 31   | 923051S     | ALR HANGER ASSM W/PAD ALUM LB         | PAIR    |

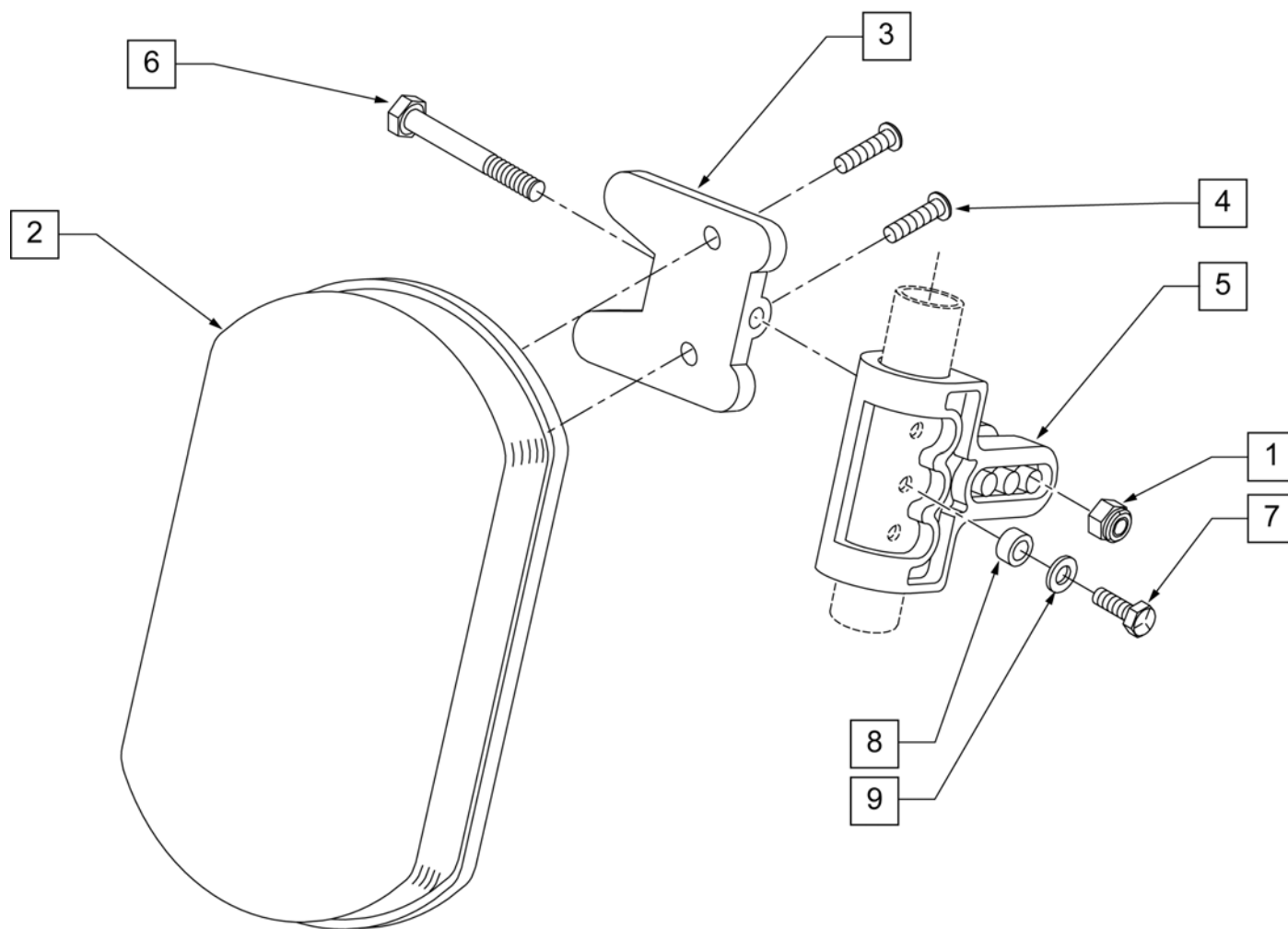


| Pos.  | Item Number | Description                   |
|-------|-------------|-------------------------------|
| 1     | 106936      | 5/16-18 NYLOCK NUT THICK      |
| 2     | 755010      | ALR/ELR PAD                   |
| 3     | 561096      | ALR/ELR CALF PAD BRACKET      |
| 4     | 281182      | 1/4-20 x 5/8 PAN PHIL         |
| 5     | 561099      | CALF CLAMP ALR                |
| 6     | 107019      | HHCS 5/16-18 X 3-1/2 ROHS     |
| 7     | 107017      | 1/4-20 x 1 1/4 G8 HEX C/S     |
| 8     | 483027      | 1/4 x 1/4 ALUMINUM SPACER BLK |
| 9     | 104414      | WAVE WASHER 6 MM INORG        |
| 1-6   | 881399      | ALR CALF PAD W/BRACKET        |
| 1,3-9 | 881398      | ALR CALF CLAMP ASSM W/O PAD   |
| 1-9   | 881400      | ALR CALF PAD ASSEMBLY (EA)    |

**NOTE:** (Discontinued on Quickie 2 for 60° and 70° Hangers 10/23/99- Parts only).

**NOTE:** Used on Articulating Hanger Assemblies.

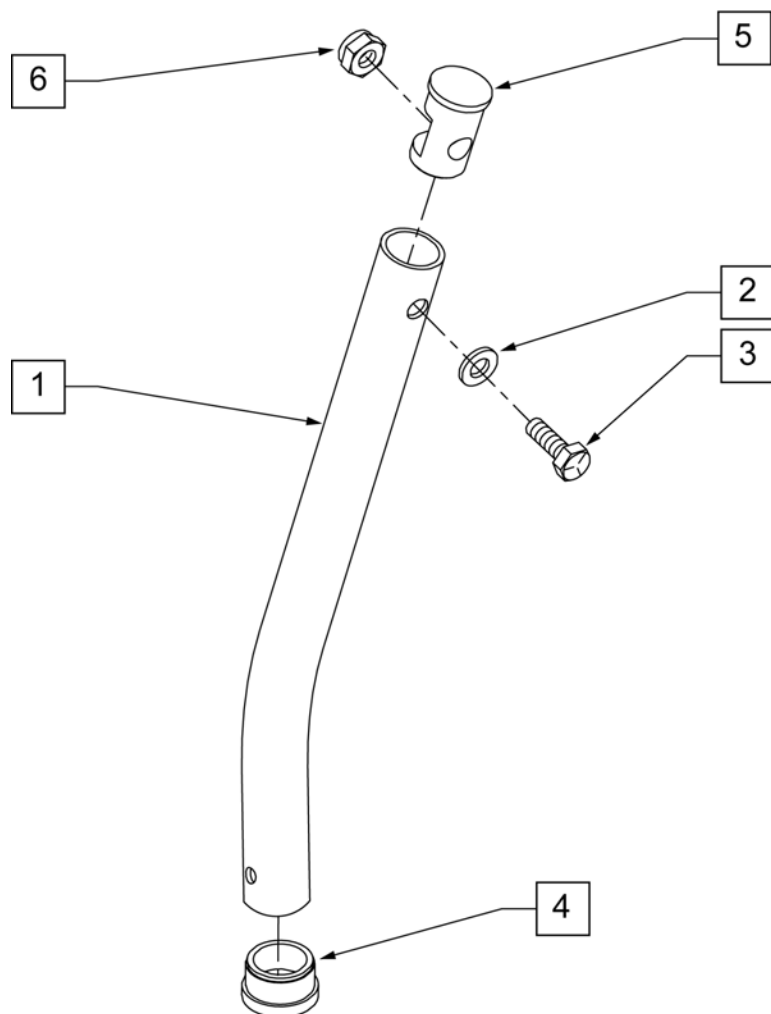


MEASURING GUIDE

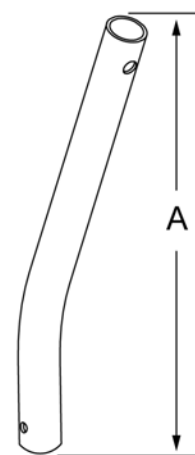

|        | "A"     |
|--------|---------|
| SHORT  | 6 5/8"  |
| MED    | 9"      |
| LONG   | 11 5/8" |
| X-LONG | 13 1/2" |

| Pos. | Item Number | Description                           |
|------|-------------|---------------------------------------|
| 1    | 521010      | FOOTREST EXT TUBE ONLY SHT BLK        |
| 1    | 521011      | FOOTREST EXT TUBE ONLY MED BLK        |
| 1    | 521012      | FOOTREST EXT TUBE ONLY LNG BLK        |
| 1    | 521013      | FOOTREST EXT TUBE ONLY X-LNG BLK      |
| 2    | 104414      | WAVE WASHER 6 MM INORG                |
| 3    | 106958      | HHCS 1/4-20 X 1 ROHS                  |
| 4    | 561000-QU   | 3/4" INSERT PLUG .083                 |
| 5    | 483553      | FOOTREST INSERT PLUG Q2               |
| 6    | 106932      | NUT NYLOCK 1/4-20 ROHS                |
| 1-6  | 921221      | FOOTREST EXT TUBE SHT ASSM BLK (EA)   |
| 1-6  | 921222      | FOOTREST EXT TUBE MED ASSM BLK (EA)   |
| 1-6  | 921223      | FOOTREST EXT TUBE LNG ASSM BLK (EA)   |
| 1-6  | 921224      | FOOTREST EXT TUBE X-LNG ASSM BLK (EA) |

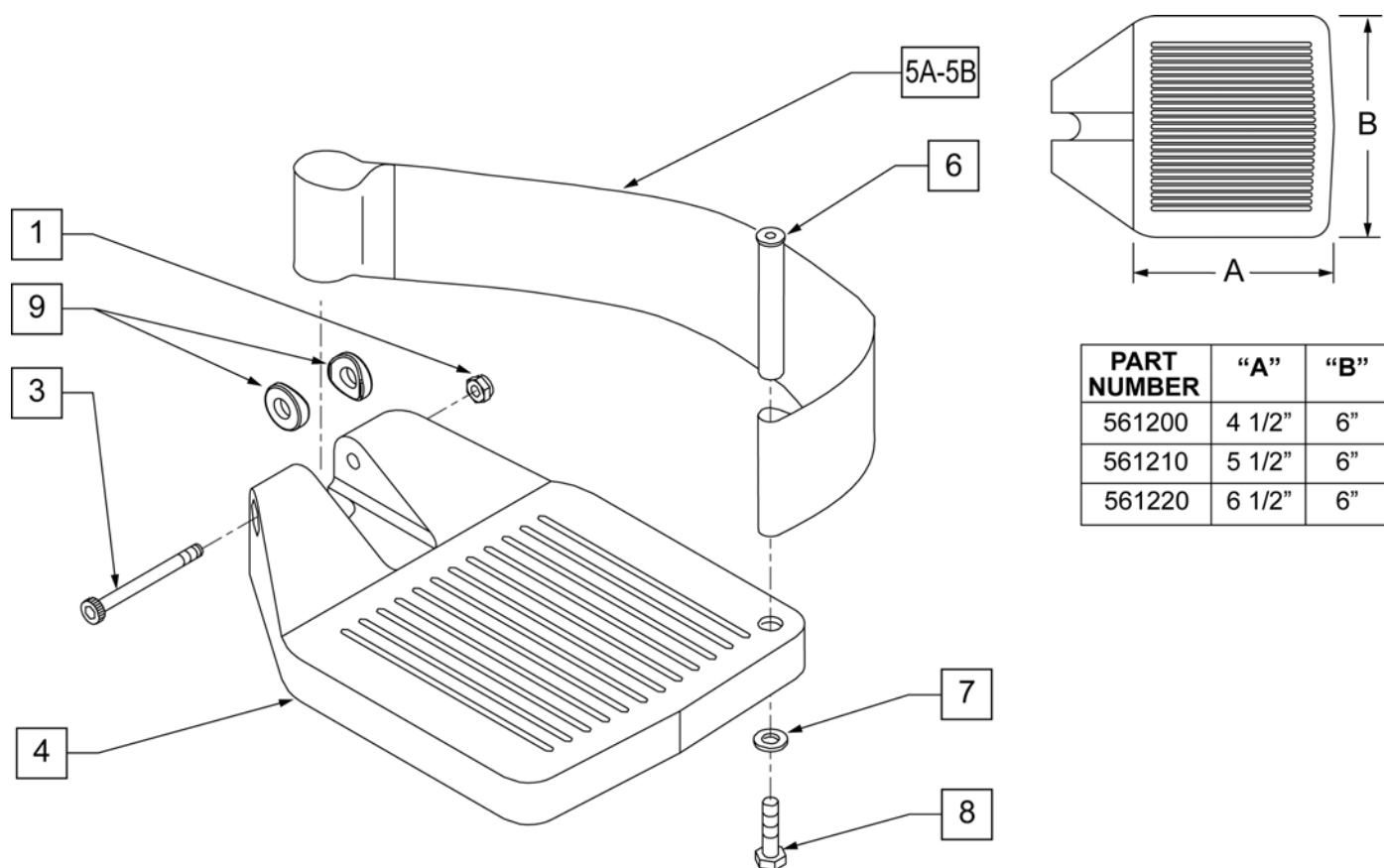

| Pos.  | Item Number | Description                                   | Remarks                                 |
|-------|-------------|-----------------------------------------------|-----------------------------------------|
| 1     | 106935      | 1/4 - 28 NYLOCK NUT                           |                                         |
| 3     | 226165      | 1/4 - 28 x 2 1/2 SOC C/S                      |                                         |
| 4     | 561200      | COMPOSITE FOOTPLATE 14 -15 (16 -18 V )        | 14 -15 OR 16 -18V                       |
| 4     | 561210      | COMPOSITE FOOTPLATE 16 -17 (19 -20 V )        | 16 -17 OR 19 -20V                       |
| 4     | 561220      | COMPOSITE FOOTPLATE 18 -20                    | 18 -20                                  |
| 5A    | 741151      | HEEL LOOP 14-15 COMPOSITE (14 -15 or 16 -18V) | 14 -15 OR 16 -18V                       |
| 5A    | 741156      | HEEL LOOP 16 -17 or 19 -20V                   | 16 -17 OR 19 -20V                       |
| 5A    | 741153      | HEEL LOOP 18-20 COMPOSITE/14-15 ANG ADJ       | 18 -20                                  |
| 5A    | 902270      | HEEL LOOP COMPOSITE FT 14-15                  | PAIR 14 -15 OR 16 -18V                  |
| 5A    | 902271      | HEEL LOOP COMPOSITE FT 16-17                  | PAIR 16 -17 OR 19 -20V                  |
| 5A    | 902272      | HEEL LOOP COMPOSITE FT 18-20                  | PAIR 18 -20                             |
| 5B    | 741154      | HEEL LOOP 14-15 COMPOSITE HEMI                | 14 -15 HEMI                             |
| 5B    | 741155      | HEEL LOOP 16-17 COMPOSITE HEMI                | 16 -17 HEMI                             |
| 5B    | 741153      | HEEL LOOP 18-20 COMPOSITE HEMI                | 18 -10 HEMI                             |
| 5B    | 902260      | HEEL LOOPS HEMI COMPOSITE 14-15               | PAIR 14 -15 HEMI                        |
| 5B    | 902261      | HEEL LOOPS HEMI COMPOSITE 16-17               | PAIR 16 -17 HEMI                        |
| 5B    | 902262      | HEEL LOOPS HEMI COMPOSITE 18-20               | PAIR 18 -20 HEMI                        |
| 6     | 561040      | HEEL LOOP PEG                                 |                                         |
| 7     | 106944      | 1/4 AN WASHER FLAT THIN                       |                                         |
| 8     | 106958      | HHCS 1/4-20 X 1 ROHS                          |                                         |
| 9     | 561003      | 3/4" SADDLE THIN                              | USED W/ ROUND EXT TUBES ONLY            |
| 1,3,4 | 921180      | COMPOSITE FOOTPLATE ASSM 14-15                | RIGHT (14 -15 OR 16 -18V) W/O HEEL LOOP |
| 1,3,4 | 921181      | COMPOSITE FOOTPLATE ASSM 14-15                | LEFT (14 -15 OR 16 -18V) W/O HEEL LOOP  |
| 1,3,4 | 921182      | COMPOSITE FOOTPLATE ASSM 16-17                | RIGHT (16 -17 OR 19 -20V) W/O HEEL LOOP |
| 1,3,4 | 921183      | COMPOSITE FOOTPLATE ASSM 16-17                | LEFT (16 -17 OR 19 -20V) W/O HEEL LOOP  |

| Pos.     | Item Number | Description                                | Remarks                                |
|----------|-------------|--------------------------------------------|----------------------------------------|
| 1,3,4    | 921184      | COMPOSITE FOOTPLATE ASSM 18-20             | RIGHT (18 -20) W/O HEEL LOOP           |
| 1,3,4    | 921185      | COMPOSITE FOOTPLATE ASSM 18-20             | LEFT (18 -20) W/O HEEL LOOP            |
| 1,3,4    | 921151      | COMPOSITE FOOTPLATE ASSM 14-15/16-18V      | PAIR (14 -15 OR 16 -18V) W/O HEEL LOOP |
| 1,3,4    | 921152      | COMPOSITE FOOTPLATE ASSM 16-17/19-20V      | PAIR (16 -17 OR 19 -20V) W/O HEEL LOOP |
| 1,3,4    | 921153      | COMPOSITE FOOTPLATE ASSM 18-20             | PAIR (18 -20) W/O HEEL LOOP            |
| 1,3,4A-8 | 921190      | COMPOSITE FOOTPLATE ASSM HL LP 14-15       | RIGHT (14 -15 OR 16 -18V) W/ HEEL LOOP |
| 1,3,4A-8 | 921191      | COMPOSITE FOOTPLATE ASSM HL LP 14-15       | LEFT (14 -15 OR 16 -18V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921192      | COMPOSITE FOOTPLATE ASSM HL LP 16-17       | RIGHT (16 -17 OR 19 -20V) W/ HEEL LOOP |
| 1,3,4A-8 | 921193      | COMPOSITE FOOTPLATE ASSM HL LP 16-17       | LEFT (16 -17 OR 19 -20V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921194      | COMPOSITE FOOTPLATE ASSM HL LP 18-20       | RIGHT (18 -20) W/ HEEL LOOP            |
| 1,3,4A-8 | 921195      | COMPOSITE FOOTPLATE ASSM HL LP 18-20       | LEFT (18 -20) W/ HEEL LOOP             |
| 1,3,4A-8 | 921161      | COMPOSITE FOOTPLATE ASSM HL 14-15/16-18V   | PAIR (14 -15 OR 16 -18V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921162      | COMPOSITE FOOTPLATE ASSM HL 16-17/19-20V   | PAIR (16 -17 OR 19 -20V) W/ HEEL LOOP  |
| 1,3,4A-8 | 921163      | COMPOSITE FOOTPLATE ASSM HL 18-20          | PAIR (18 -20) W/ HEEL LOOP             |
| 1,3,4B-8 | 921140      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15  | RIGHT (14 -15) W/ HEEL LOOP            |
| 1,3,4B-8 | 921141      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15  | LEFT (14 -15) W/ HEEL LOOP             |
| 1,3,4B-8 | 921142      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17  | RIGHT (16 -17) W/ HEEL LOOP            |
| 1,3,4B-8 | 921143      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17  | LEFT (16 -17) W/ HEEL LOOP             |
| 1,3,4B-8 | 921144      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20  | RIGHT (18 -20) W/ HEEL LOOP            |
| 1,3,4B-8 | 921145      | COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20  | LEFT (18 -20) W/ HEEL LOOP             |
| 1,3,4B-8 | 921167      | COMPOSITE FOOTPLATE ASSM HEMI HL LPS 14-15 | PAIR (14 -15) W/ HEEL LOOP             |
| 1,3,4B-8 | 921168      | COMPOSITE FOOTPLATE ASSM HEMI HL LPS 16-17 | PAIR (16 -17) W/ HEEL LOOP             |
| 1,3,4B-8 | 921169      | COMPOSITE FOOTPLATE ASSM HEMI HL LPS 18-20 | PAIR (18 -20) W/ HEEL LOOP             |




**NOTE:** FOR HIGH-MOUNT ADULT ANGLE-ADJUSTABLE FOOTPLATE, SEE "HIGH-MOUNT EXTENSION TUBE" PAGE.

**NOTE:** ORDER ITEM 4B FOR CHAIRS PRIOR TO 8/13/01.

**NOTE:** ONLY 1 WASHER PRIOR TO 8/13/01.

**NOTE:** ORDER PART 226167 PRIOR TO 8/13/01.

**NOTE:** See attached REFERENCE GUIDE.

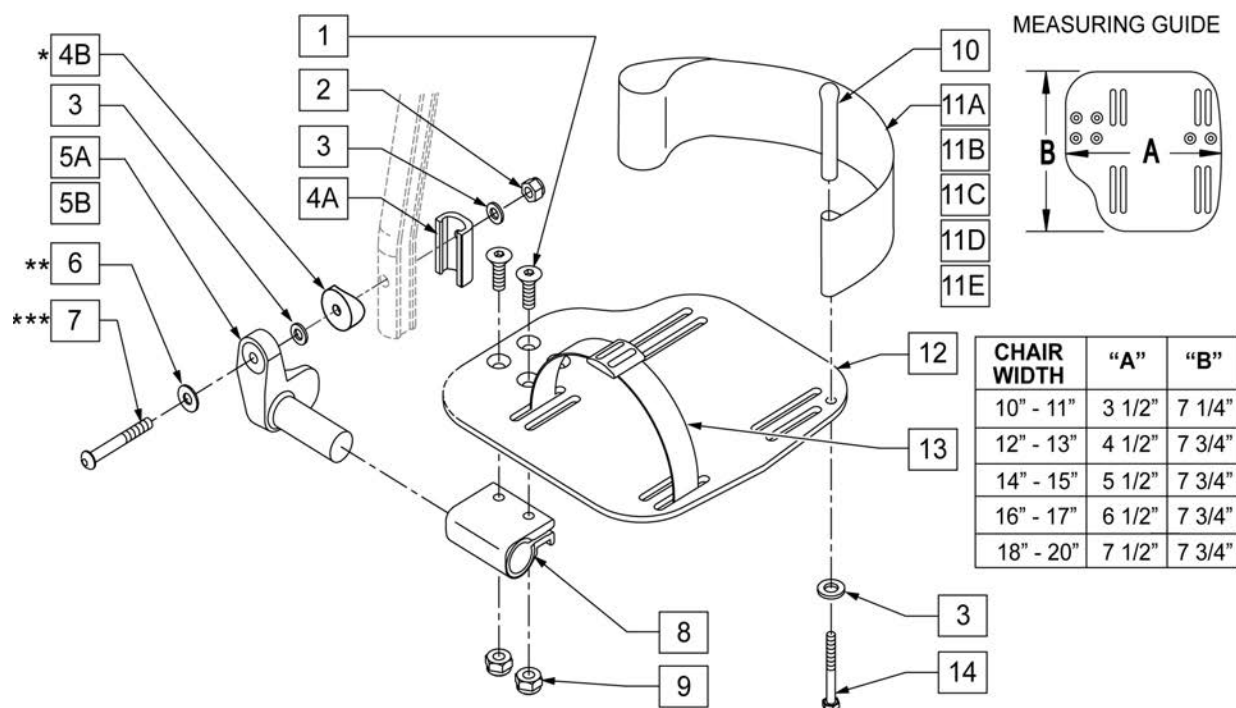


| Pos. | Item Number | Description           | Remarks |
|------|-------------|-----------------------|---------|
| 1    | 106978      | 1/4-28 x 7/8 FH C/S   |         |
| 2    | 106941      | 1/4-28 AUTOMATION NUT |         |

| Pos.                    | Item Number | Description                         | Remarks                                    |
|-------------------------|-------------|-------------------------------------|--------------------------------------------|
| 3                       | 106943      | WASHER FLAT AN 1/4 ROHS             |                                            |
| 4A                      | 483720      | HANGER EXTENSION SADDLE LONG        | USE W/ DOUBLE D EXT TUBE                   |
| 4B                      | 561055      | 3/4" SADDLE 1" OD                   | USE W/ ROUND EXT TUBE                      |
| 5                       | 541002      | FTRST MOUNTING BRKT-3/8 RAD R       | RIGHT                                      |
| 5                       | 541003      | FTRST MOUNTING BRKT-3/8 RAD L       | LEFT                                       |
| 6                       | 106945      | WASHER FLAT SAE 1/4 ROHS            |                                            |
| 7A                      | 106962      | BHCS 1/4-28 X 1-3/4 ROHS            | USE W/ DOUBLE D EXT TUBE                   |
| 7B                      | 226167      | 1/4-28 x 1.625 BH C/S BLK           | USE W/ ROUND EXT TUBE                      |
| 8                       | 550350      | 3/4" CLMP ADJ FT PLT                |                                            |
| 9                       | 106935      | NUT NYLOCK THIN 1/4-28 ROHS         |                                            |
| 10                      | 561040      | HEEL LOOP PEG                       |                                            |
| 11A                     | 741154      | HEEL LOOP 14-15 CMPST HEMI          | USED W/ 10-11" ADULT ANG ADJ FTPLTS        |
| 11B                     | 741155      | HEEL LOOP 16-17 CMPST HEMI          | USED W/ 12 -13" ADULT ANG ADJ FTPLTS       |
| 11C                     | 741153      | H/L 18-20 CMPST/14-15 ANG ADJ       | USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS |
| 11D                     | 741253      | HEEL LOOP ADLT ADJ F/PLT 18-20      | USED W/ 18 -20" ADULT ANG ADJ FTPLTS       |
| 12                      | 500656      | ADLT ANGL ADJ F/PLT 10/11" R        | RIGHT                                      |
| 12                      | 500657      | ADLT ANGL ADJ F/PLT 10/11" L        | LEFT                                       |
| 12                      | 500658      | ADLT ANGL ADJ F/PLT 12/13" R        | RIGHT                                      |
| 12                      | 500659      | ADLT ANGL ADJ F/PLT 12/13" L        | LEFT                                       |
| 12                      | 500660      | ADLT ANGL ADJ F/PLT 14/15           | RIGHT                                      |
| 12                      | 500661      | ADLT ANGL ADJ F/PLT 14/15           | LEFT                                       |
| 12                      | 500662      | ADLT ANGL ADJ F/PLT 16/17           | RIGHT                                      |
| 12                      | 500663      | ADLT ANGL ADJ F/PLT 16/17           | LEFT                                       |
| 12                      | 500664      | ADLT ANGL ADJ F/PLT 18-20           | RIGHT                                      |
| 12                      | 500665      | ADLT ANGL ADJ F/PLT 18-20           | LEFT                                       |
| 13                      | 740260      | TOE/ANKLE STRAP ADJ ADLT F/PLT      |                                            |
| 14                      | 106958      | 1/4-20 x 1 HEX C/S                  |                                            |
| 1-3,5,6,8,9,12          | 820526      | AD ANG ADJ F/PLT R 12"/13" OMIT HL  | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820536      | AD ANG ADJ F/PLT L 12"/13" OMT HL   | LEFT                                       |
| 1-3,5,6,8,9,12          | 820460      | FP ADLT ADJ 14/15 OMIT HL           | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820462      | FP ADLT ADJ 14/15 OMIT HL           | LEFT                                       |
| 1-3,5,6,8,9,12          | 820464      | FP ADLT ADJ 16/17 OMIT HL           | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820466      | FP ADLT ADJ 16/17 OMIT HL           | LEFT                                       |
| 1-3,5,6,8,9,12          | 820468      | FP ADLT ADJ 18-20 OMIT HL           | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820470      | FP ADLT ADJ 18-20 OMIT HL           | LEFT                                       |
| 1-3,5,6,8,9,12          | 921466      | ADLT ANGL ADJ F/PLT 12"/13" OMIT HL | PAIR                                       |
| 1-3,5,6,8,9,12          | 921320      | FP ADLT ADJ 14/15 OMIT HL           | PAIR                                       |
| 1-3,5,6,8,9,12          | 921322      | FP ADLT ADJ 16/17 OMIT HL           | PAIR                                       |
| 1-3,5,6,8,9,12          | 921324      | FP ADLT ADJ 18-20 OMIT HL           | PAIR                                       |
| 1-3,5,6,8,9,12          | 820522      | AD ANGL ADJ F/PLT R 12/13" H/L      | RIGHT                                      |
| 1-3,5,6,8,9,12          | 820532      | AD ANGL ADJ F/PLT L 12/13" H/L      | LEFT                                       |
| 1-3,5,6,8-10,11C, 12,14 | 820461      | FP ADLT ADJ 14/15 HL                | RIGHT                                      |
| 1-3,5,6,8-10,11C, 12,14 | 820463      | FP ADLT ADJ 14/15 HL                | LEFT                                       |
| 1-3,5,6,8-10,11C, 12,14 | 820465      | FP ADLT ADJ 16/17 HL                | RIGHT                                      |
| 1-3,5,6,8-10,11C, 12,14 | 820467      | FP ADLT ADJ 16/17 HL                | LEFT                                       |
| 1-3,5,6,8-10,11D, 12,14 | 820469      | FP ADLT ADJ 18-20 HL                | RIGHT                                      |
| 1-3,5,6,8-10,11D, 12,14 | 820471      | FP ADLT ADJ 18-20 HL                | LEFT                                       |
| 1-3,5,6,8-10,11B, 12,14 | 921462      | ADLT ANGL ADJ F/PLATE 12"/13" HL    | PAIR                                       |
| 1-3,5,6,8-10,11C, 12,14 | 921321      | FP ADLT ADJ 14/15 HL                | PAIR                                       |
| 1-3,5,6,8-10,11C, 12,14 | 921323      | FP ADLT ADJ 16/17 HL                | PAIR                                       |
| 1-3,5,6,8-10,11D, 12,14 | 921325      | FP ADLT ADJ 18-20 HL                | PAIR                                       |
| 1-3,6,7A,9-10,14        | 101533      | ADLT ANG ADJ FP D2 HDWR KIT         | FOR DOUBLE D EXT TUBES                     |
| 1-3B,4B,6,7B,9,10,14    | 101534      | ADLT ANG ADJ FP RND HDWR KIT        | FOR ROUND EXT TUBES (Q2 HP ONLY)           |

**NOTE:** \* ORDER ITEM 8 PRIOR TO 8/13/01.


\*\* ORDER ITEM 5 PART 226167 PRIOR TO 8/13/01.

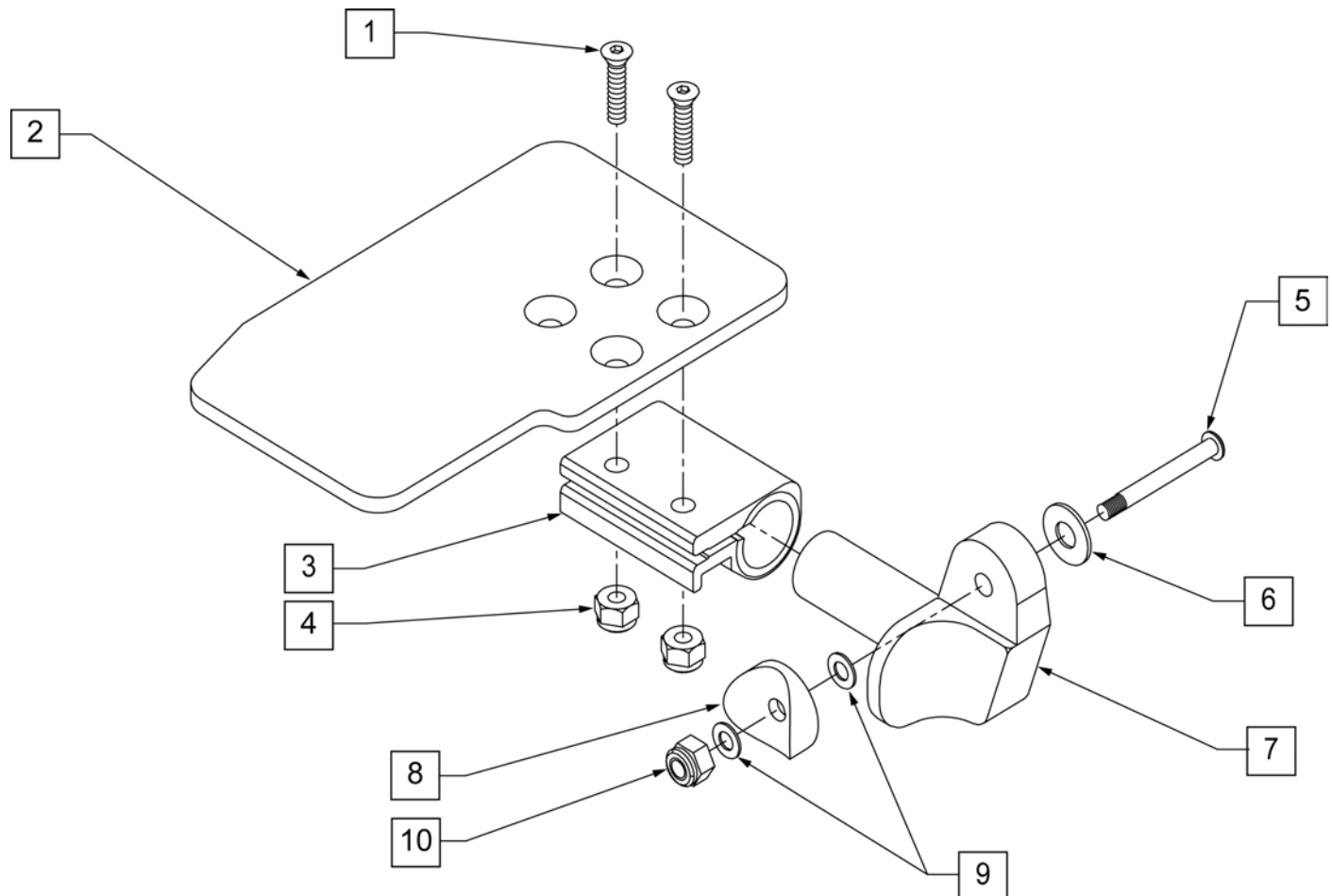

| Pos. | Item Number | Description                    | Remarks |
|------|-------------|--------------------------------|---------|
| 1    | 106978      | 1/4-28 X 7/8 FH C/S            |         |
| 2    | 500635      | KIDS ANGLE ADJ FOOTPLATE 10" R |         |
| 2    | 500636      | KIDS ANGLE ADJ FOOTPLATE 10" L |         |
| 2    | 500637      | KIDS ANGLE ADJ FOOTPLATE 11" R |         |
| 2    | 500638      | KIDS ANGLE ADJ FOOTPLATE 11" L |         |
| 2    | 500639      | KIDS ANGLE ADJ FOOTPLATE 12" R |         |
| 2    | 500640      | KIDS ANGLE ADJ FOOTPLATE 12" L |         |
| 2    | 500641      | KIDS ANGLE ADJ FOOTPLATE 13" R |         |
| 2    | 500642      | KIDS ANGLE ADJ FOOTPLATE 13" L |         |
| 2    | 500643      | KIDS ANGLE ADJ FOOTPLATE 14" R |         |
| 2    | 500644      | KIDS ANGLE ADJ FOOTPLATE 14" L |         |
| 2    | 500645      | KIDS ANGLE ADJ FOOTPLATE 15" R |         |
| 2    | 500646      | KIDS ANGLE ADJ FOOTPLATE 15" L |         |
| 2    | 500647      | KIDS ANGLE ADJ FOOTPLATE 16" R |         |
| 2    | 500648      | KIDS ANGLE ADJ FOOTPLATE 16" L |         |
| 3    | 550350      | 3/4" CLAMP ADJ FOOTPLATE       |         |
| 4    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS    |         |


| Pos.     | Item Number | Description                             | Remarks |
|----------|-------------|-----------------------------------------|---------|
| 5**      | 106962      | 1/4-28 X 1.85 BH C/S BLK ZN             |         |
| 6        | 106945      | 1/4 SAE WASHER                          |         |
| 7        | 541002      | FOOTREST MOUNTING BRACKET - 3/8" RADIUS | RIGHT   |
| 7        | 541003      | FOOTREST MOUNTING BRACKET - 3/8" RADIUS | LEFT    |
| 8*       | 561055      | 3/4" SADDLE 1" OD                       |         |
| 9        | 106944      | 1/4 AN WASHER FLAT THIN                 |         |
| 10       | 106941      | 1/4-28 AUTOMATION NUT                   |         |
| 1-7,9,10 | 828850      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 10" R  | RIGHT   |
| 1-7,9,10 | 828851      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 10" L  | LEFT    |
| 1-7,9,10 | 828852      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 11" R  | RIGHT   |
| 1-7,9,10 | 828853      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 11" L  | LEFT    |
| 1-7,9,10 | 828854      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 12" R  | RIGHT   |
| 1-7,9,10 | 828855      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 12" L  | LEFT    |
| 1-7,9,10 | 828856      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 13" R  | RIGHT   |
| 1-7,9,10 | 828857      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 13" L  | LEFT    |
| 1-7,9,10 | 828858      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 14" R  | RIGHT   |
| 1-7,9,10 | 828859      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 14" L  | LEFT    |
| 1-7,9,10 | 828860      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 15" R  | RIGHT   |
| 1-7,9,10 | 828861      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 15" L  | LEFT    |
| 1-7,9,10 | 828862      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 16" R  | RIGHT   |
| 1-7,9,10 | 828863      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 16" L  | LEFT    |
| 1-7,9,10 | 921280      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 10"    | PAIR    |
| 1-7,9,10 | 921281      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 11"    | PAIR    |
| 1-7,9,10 | 921282      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 12"    | PAIR    |
| 1-7,9,10 | 921283      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 13"    | PAIR    |
| 1-7,9,10 | 921284      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 14"    | PAIR    |
| 1-7,9,10 | 921285      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 15"    | PAIR    |
| 1-7,9,10 | 921286      | KIDS ANGLE ADJ FOOTPLATE EXT MNT 16"    | PAIR    |

**NOTE:** WHEN USING AN ARTICULATING LEGREST ASSEMBLY, THE PULL-TO-LOCK WHEEL LOCK MUST BE USED.

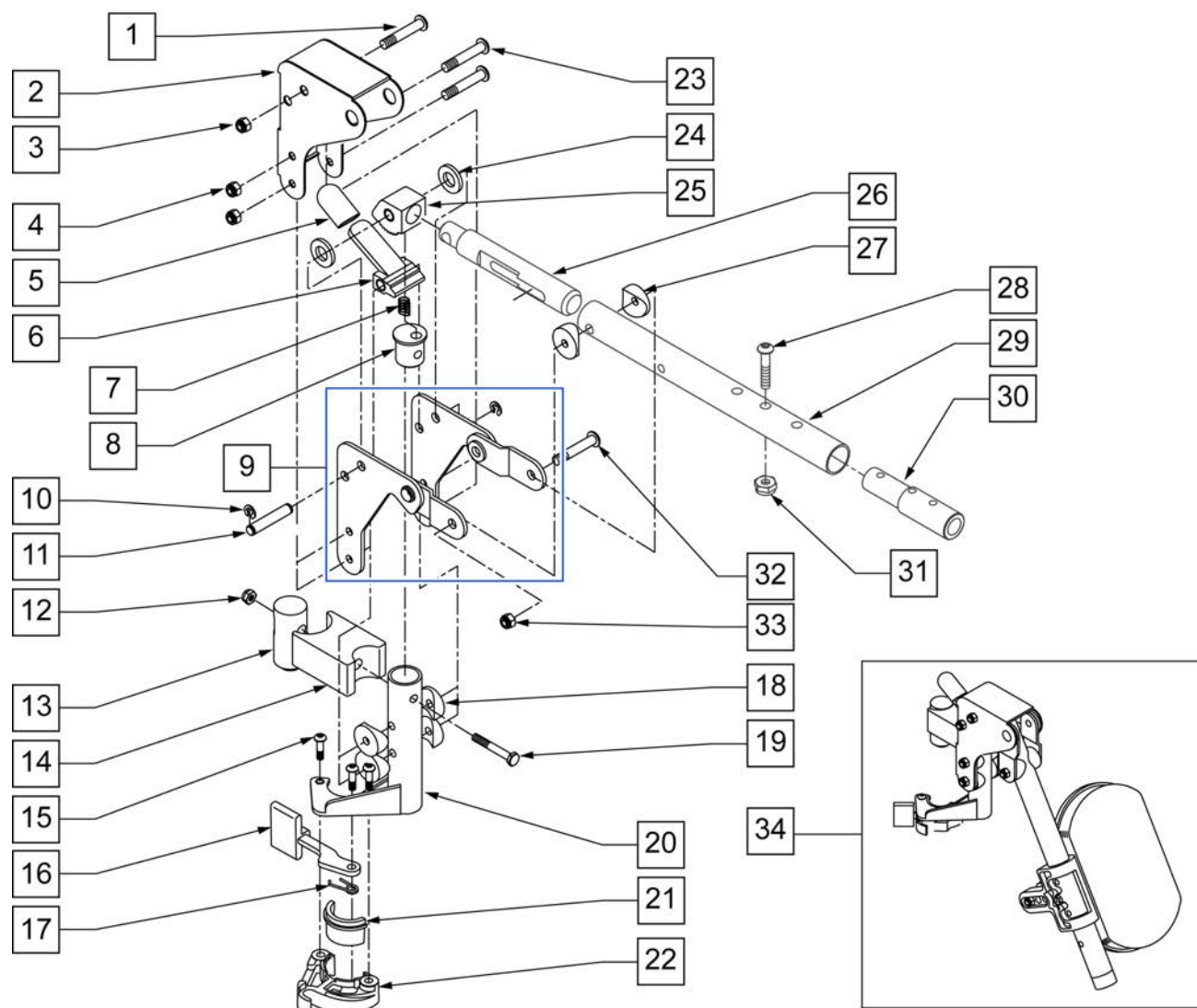


| Pos. | Item Number | Description                   | Remarks |
|------|-------------|-------------------------------|---------|
| 1    | 106983      | 5/16-18 x 2 BH C/S SHT THREAD |         |
| 2    | 562720      | ALR COVER STD                 |         |
| 3    | 106937      | 5/16-18 NYLOCK NUT THIN       |         |
| 4    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS   |         |
| 5    | 560501      | ALR RELEASE LEVER COVER       |         |
| 6    | 505902      | ALR RELEASE LEVER             |         |
| 7    | 321141      | 650L x 6 COIL SPRING          |         |
| 8    | 483060      | ALR SPRING PLUG               |         |
| 9    | 880526      | ALR BRACKET W/LINK R          |         |
| 9    | 880527      | ALR BRACKET W/LINK L          |         |
| 10   | 342000      | 1/4" E-CLIP                   |         |
| 11   | 481052      | ALR RETAINING STUD            |         |
| 12   | 106934      | 1/4-28 NYLOCK NUT             |         |

| Pos.     | Item Number | Description                    | Remarks              |
|----------|-------------|--------------------------------|----------------------|
| 13       | 500177      | ALR PIVOT SADDLE .125          |                      |
| 14       | 500178      | ALR PIVOT SADDLE BLOCK         |                      |
| 15       | 226057      | 10/24 x .725 BTTN HD SOC BZ    |                      |
| 16       | 561079      | S/A LATCH LEVER                |                      |
| 17       | 321136      | S/A SPRING R CLEAR             |                      |
| 17       | 321137      | S/A SPRING R                   |                      |
| 18       | 561004      | 1' SADDLE 1/4 HOLE             |                      |
| 19       | 107016      | 1/4-28 x 3 G8 HEX C/S          |                      |
| 20       | 882466      | ALR S/A TUBE ASSM L            |                      |
| 20       | 882467      | ALR S/A TUBE ASSM R            |                      |
| 23       | 106964      | 1/4-28 x 2 BH C/S              |                      |
| 24       | 561016      | 1" WASHER 5/16 HOLE            |                      |
| 25       | 505900      | ALR RATCHET BLOCK              |                      |
| 26       | 483062      | ALR RATCHET ROD                |                      |
| 27       | 561006      | 1" SADDLE 5/16" HOLE           |                      |
| 28       | 106961      | 1/4-28 x 1 1/4 BH C/S          |                      |
| 29       | 505025S     | KIDS ALR LEGREST TUBE 11-13"   |                      |
| 30       | 500150      | 2" FOOTREST EXT TUBE           |                      |
| 31       | 106935      | NUT NYLOCK THIN 1/4-28 ROHS    |                      |
| 32       | 226195      | 5/16-18 x 1.82 BH C/S          |                      |
| 33       | 106937      | 5/16-18 NYLOCK NUT THIN        |                      |
| 34       | 923053S     | KIDS ALR HANGER ASSEM W/PAD    | (PR) (SPECIFY COLOR) |
| 28,30,31 | 111505      | 2" FTREST EXT W/ HDWR          |                      |
| 21,22    | 881500      | LATCH BLOCK ASSM R             |                      |
| 21,22    | 881501      | LATCH BLOCK ASSM L             |                      |
| 1-31     | 641432S     | KIDS ALR HNGR W/O PAD R 2" EXT |                      |
| 1-31     | 641433S     | KIDS ALR HNGR W/O PAD L 2" EXT |                      |

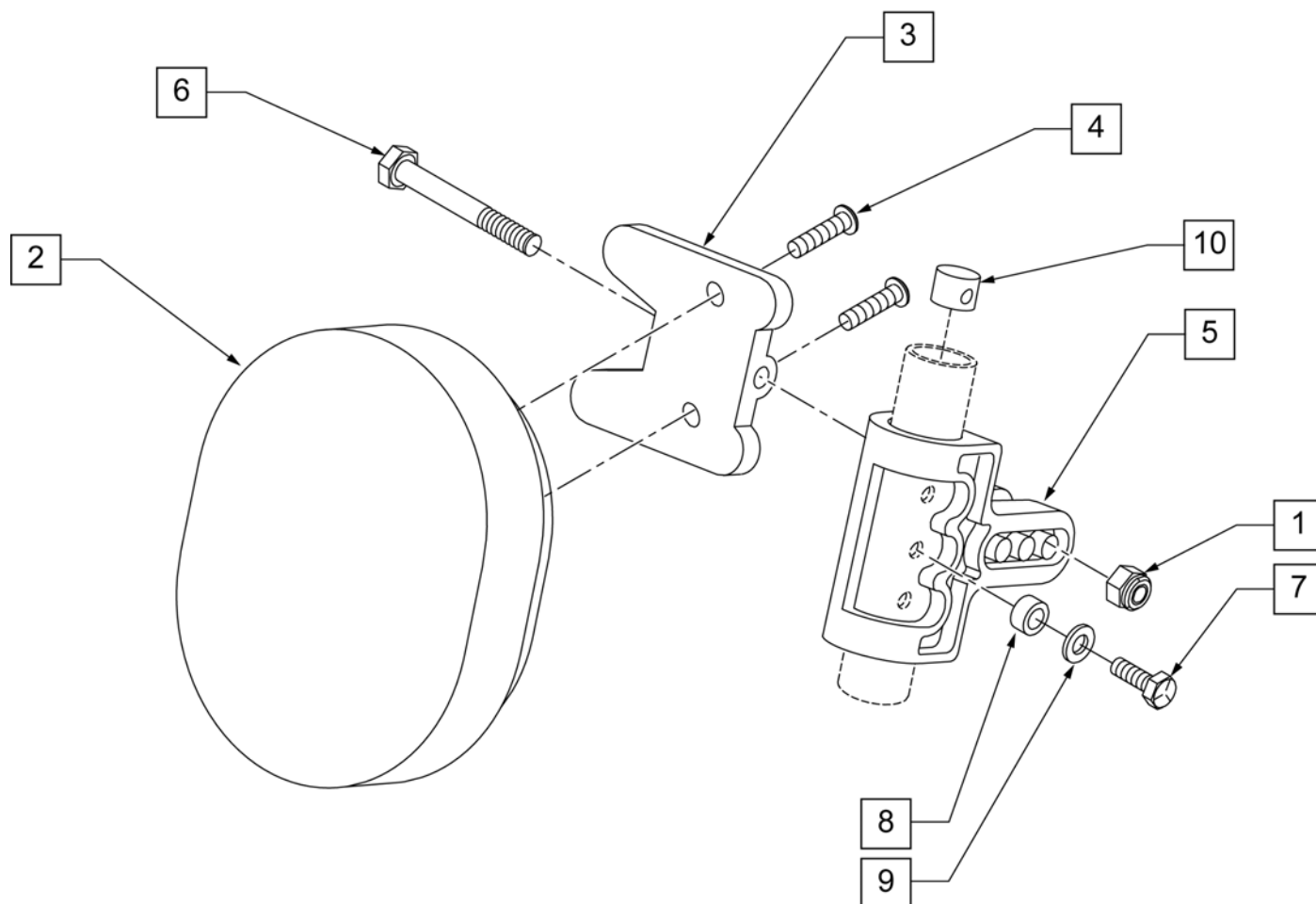


| Pos. | Item Number | Description                     |
|------|-------------|---------------------------------|
| 1    | 106936      | 5/16-18 NYLOCK NUT THICK        |
| 2    | 755020      | 4 x 5 OVAL PAD                  |
| 3    | 561096      | ALR/ELR CALF PAD BRACKET        |
| 4    | 281181      | 1/4-20 x 3/4 PAN PHIL           |
| 5    | 561099      | CALF CLAMP ALR                  |
| 6    | 107019      | HHCS 5/16-18 X 3-1/2 ROHS       |
| 7    | 107017      | 1/4-20 x 1 1/4 G8 HEX C/S       |
| 8    | 483027      | 1/4 x 1/4 ALUMINUM SPACER       |
| 9    | 262130      | 6mm WASHER WAVE                 |
| 10   | 500151      | KIDS ALR RETAINING PLUG         |
| 1-6  | 881401      | KIDS ALR OVAL PAD W/BRACKET     |
| 1-10 | 881402      | KIDS ALR OVAL PAD ASSEMBLY (EA) |

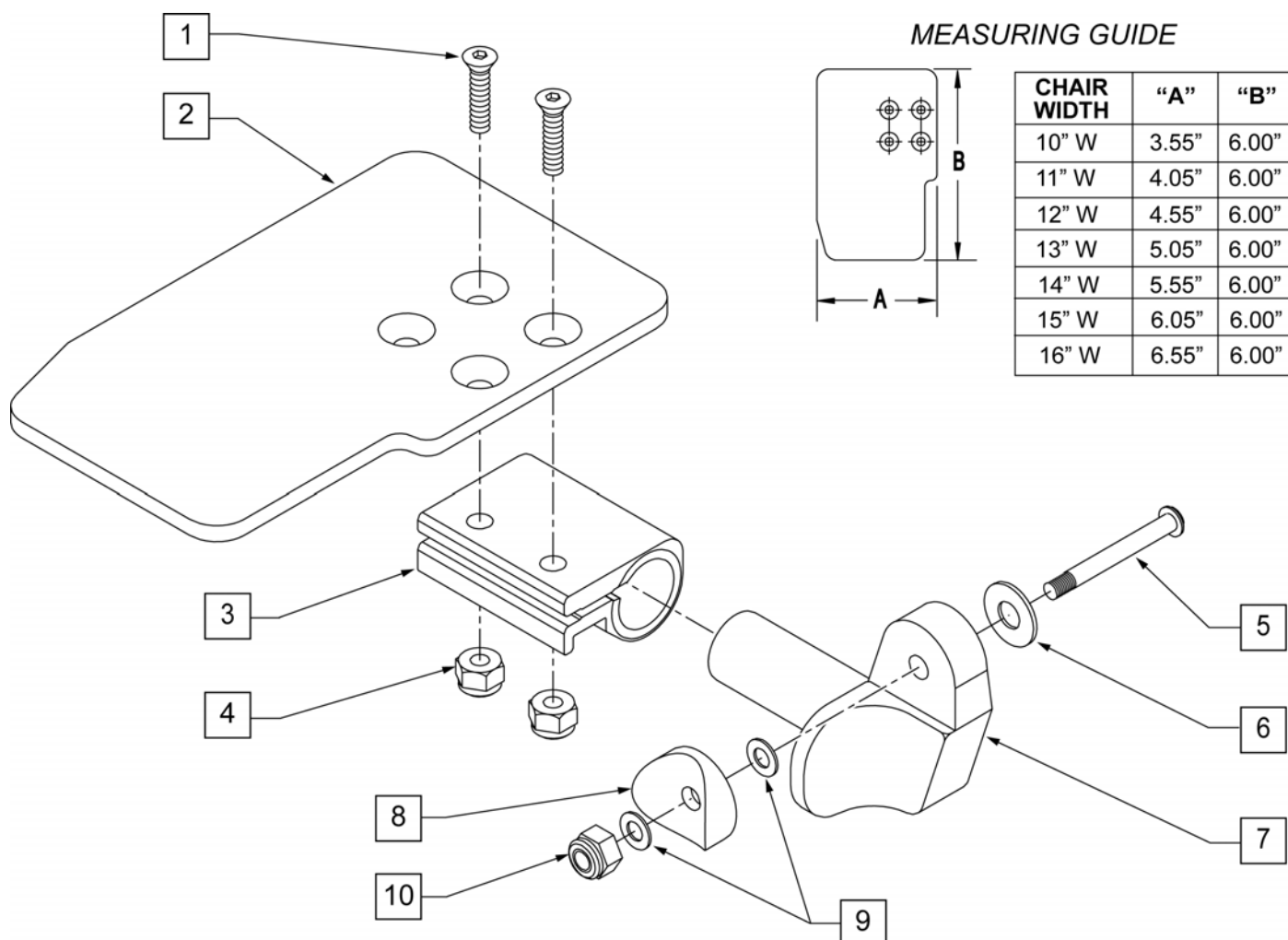

| Pos. | Item Number | Description                 | Remarks | Pos. | Item Number | Description                | Remarks |
|------|-------------|-----------------------------|---------|------|-------------|----------------------------|---------|
| 1    | 106978      | FHCS 1/4-28 X 7/8 ROHS      |         | 7    | 541001      | FTRST MOUNTING BRKT 1 RAD  | LEFT    |
| 2    | 500635      | KIDS ANGL ADJ F/PLT 10      | RIGHT   | 8    | 561004      | 1 SADDLE 1/4 HOLE          |         |
| 2    | 500636      | KIDS ANGL ADJ F/PLT 10      | LEFT    | 9    | 106943      | WASHER FLAT THIN AN 1/4    |         |
| 2    | 500637      | KIDS ANGL ADJ F/PLT 11      | RIGHT   | 10   | 106941      | NUT AUTOMATION 1/4-28 ROHS |         |
| 2    | 500638      | KIDS ANGL ADJ F/PLT 11      | LEFT    | 1-10 | 828821      | KIDS ANG ADJ F/PLT ASSM 10 | RIGHT   |
| 2    | 500639      | KIDS ANGL ADJ F/PLT 12      | RIGHT   | 1-10 | 828822      | KIDS ANG ADJ F/PLT ASSM 10 | LEFT    |
| 2    | 500640      | KIDS ANGL ADJ F/PLT 12      | LEFT    | 1-10 | 828823      | KIDS ANG ADJ F/PLT ASSM 11 | RIGHT   |
| 2    | 500641      | KIDS ANGL ADJ F/PLT 13      | RIGHT   | 1-10 | 828824      | KIDS ANG ADJ F/PLT ASSM 11 | LEFT    |
| 2    | 500642      | KIDS ANGL ADJ F/PLT 13      | LEFT    | 1-10 | 828825      | KIDS ANG ADJ F/PLT ASSM 12 | RIGHT   |
| 2    | 500643      | KIDS ANGL ADJ F/PLT 14      | RIGHT   | 1-10 | 828826      | KIDS ANG ADJ F/PLT ASSM 12 | LEFT    |
| 2    | 500644      | KIDS ANGL ADJ F/PLT 14      | LEFT    | 1-10 | 828827      | KIDS ANG ADJ F/PLT ASSM 13 | RIGHT   |
| 2    | 500645      | KIDS ANGL ADJ F/PLT 15      | RIGHT   | 1-10 | 828828      | KIDS ANG ADJ F/PLT ASSM 13 | LEFT    |
| 2    | 500646      | KIDS ANGL ADJ F/PLT 15      | LEFT    | 1-10 | 828829      | KIDS ANG ADJ F/PLT ASSM 14 | RIGHT   |
| 2    | 500647      | KIDS ANGL ADJ F/PLT 16      | RIGHT   | 1-10 | 828830      | KIDS ANG ADJ F/PLT ASSM 14 | LEFT    |
| 2    | 500648      | KIDS ANGL ADJ F/PLT 16      | LEFT    | 1-10 | 828831      | KIDS ANG ADJ F/PLT ASSM 15 | RIGHT   |
| 3    | 550350      | 3/4 CLAMP ADJ FOOTPLATE     |         | 1-10 | 828832      | KIDS ANG ADJ F/PLT ASSM 15 | LEFT    |
| 4    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS |         | 1-10 | 828833      | KIDS ANG ADJ F/PLT ASSM 16 | RIGHT   |
| 5    | 226147      | 1/4-28 X 1.85 BH C/S BLK ZN |         | 1-10 | 828834      | KIDS ANG ADJ F/PLT ASSM 16 | LEFT    |
| 6    | 106945      | WASHER FLAT SAE 1/4 ROHS    |         | 1-10 | 921226      | KIDS ANG ADJ F/PLT ASSM 10 | PAIR    |
| 7    | 541000      | FTRST MOUNTING BRKT 1 RAD   | RIGHT   | 1-10 | 921227      | KIDS ANG ADJ F/PLT ASSM 11 | PAIR    |





| Pos. | Item Number | Description                | Remarks | Pos. | Item Number | Description                | Remarks |
|------|-------------|----------------------------|---------|------|-------------|----------------------------|---------|
| 1-10 | 921228      | KIDS ANG ADJ F/PLT ASSM 12 | PAIR    | 1-10 | 921231      | KIDS ANG ADJ F/PLT ASSM 15 | PAIR    |
| 1-10 | 921229      | KIDS ANG ADJ F/PLT ASSM 13 | PAIR    | 1-10 | 921232      | KIDS ANG ADJ F/PLT ASSM 16 | PAIR    |
| 1-10 | 921230      | KIDS ANG ADJ F/PLT ASSM 14 | PAIR    |      |             |                            |         |

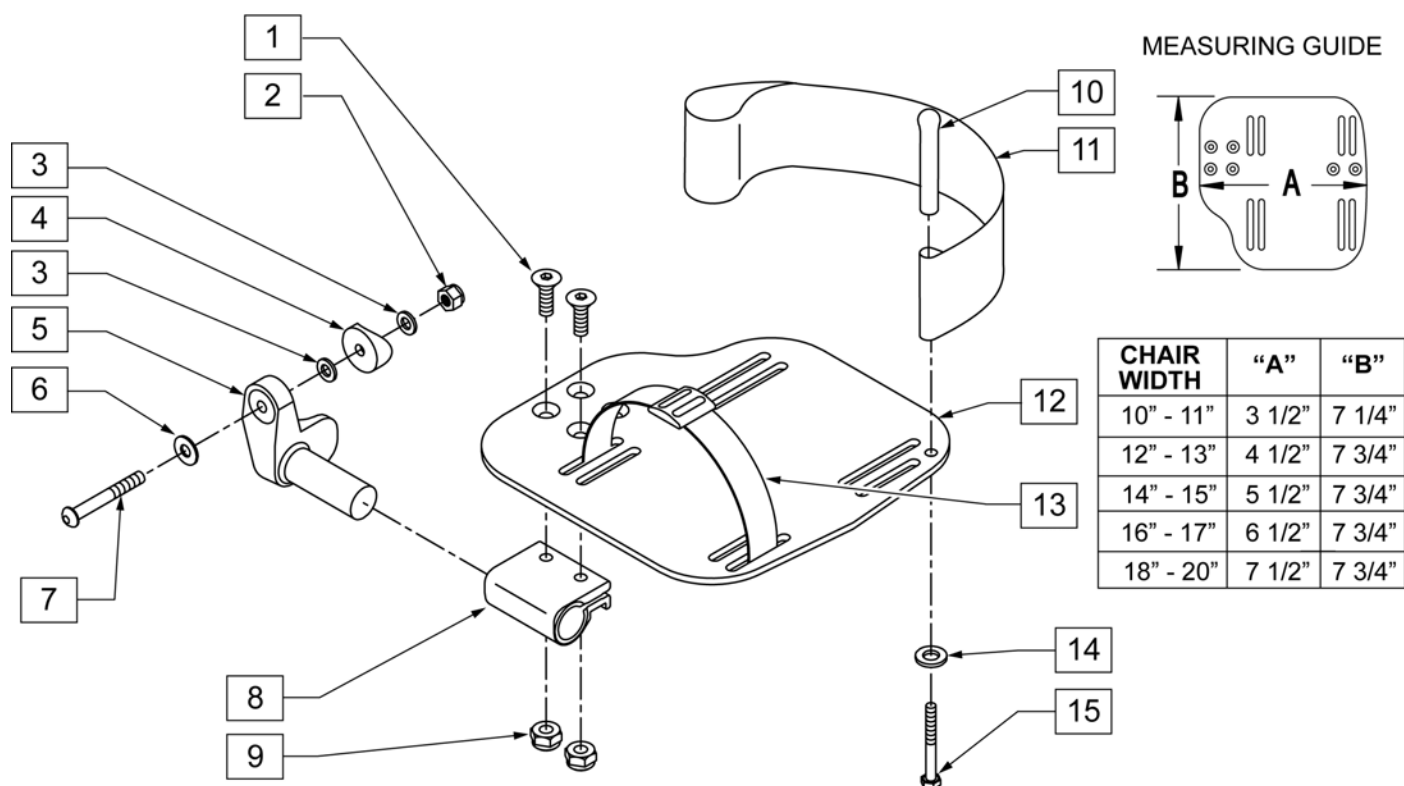


| Pos. | Item Number | Description                         | Remarks                                    |
|------|-------------|-------------------------------------|--------------------------------------------|
| 1    | 106978      | FHCS 1/4-28 X 7/8 ROHS              |                                            |
| 2    | 106941      | NUT AUTOMATION 1/4-28 ROHS          |                                            |
| 3    | 106943      | WASHER FLAT AN 1/4 ROHS             |                                            |
| 4    | 561004      | 1" SADDLE 1/4" HOLE                 |                                            |
| 5    | 541000      | FOOTREST MOUNTING BRKT-1" RAD       | RIGHT                                      |
| 5    | 541001      | FOOTREST MOUNTING BRKT-1" RAD       | LEFT                                       |
| 6    | 106945      | WASHER FLAT SAE 1/4 ROHS            |                                            |
| 7    | 106964      | BHCS 1/4-28 X 2                     |                                            |
| 8    | 550350      | 3/4" CLAMP ADJ FOOTPLATE            |                                            |
| 9    | 106935      | NUT NYLOCK THIN 1/4-28 ROHS         |                                            |
| 10   | 561040      | HEEL LOOP PEG                       |                                            |
| 11   | 741153      | HEEL LOOP 18-20 CMPST/14-15 ANG ADJ | USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS |
| 11   | 741253      | HEEL LOOP ADLT ADJ F/PLT 18-20      | USED W/ 18 -20" ADULT ANG ADJ FTPLTS       |
| 12   | 500656      | ADULT ANGL ADJ F/PLT 10/11          | RIGHT                                      |
| 12   | 500657      | ADULT ANGL ADJ F/PLT 10/11          | LEFT                                       |
| 12   | 500658      | ADULT ANGL ADJ F/PLT 12/13          | RIGHT                                      |
| 12   | 500659      | ADULT ANGL ADJ F/PLT 12/13          | LEFT                                       |
| 12   | 500660      | ADULT ANGL ADJ F/PLT 14/15          | RIGHT                                      |
| 12   | 500661      | ADULT ANGL ADJ F/PLT 14/15          | LEFT                                       |
| 12   | 500662      | ADULT ANGL ADJ F/PLT 16/17          | RIGHT                                      |
| 12   | 500663      | ADULT ANGL ADJ F/PLT 16/17          | LEFT                                       |
| 12   | 500664      | ADULT ANGL ADJ F/PLT 18-20          | RIGHT                                      |
| 12   | 500665      | ADULT ANGL ADJ F/PLT 18-20          | LEFT                                       |
| 13   | 740260      | TOE-ANKLE STRAP ADJ ADULT FOOTPLATE |                                            |
| 14   | 106943      | WASHER FLAT AN 1/4 ROHS             |                                            |

| Pos.       | Item Number | Description                       | Remarks |
|------------|-------------|-----------------------------------|---------|
| 15         | 106958      | HHCS 1/4-20 X 1 ROHS              |         |
| 1-9,12     | 820544      | FP ADULT ADJ 90D 10/11 R          | RIGHT   |
| 1-9,12     | 820554      | FP ADULT ADJ 90D 10/11 L          | LEFT    |
| 1-9,12     | 820546      | FP ADULT ADJ 90D 12/13 R          | RIGHT   |
| 1-9,12     | 820556      | FP ADULT ADJ 90D 12/13 L          | LEFT    |
| 1-9,12     | 820472      | FP ADULT ADJ 90D 14/15 R          | RIGHT   |
| 1-9,12     | 820473      | FP ADULT ADJ 90D 14/15 L          | LEFT    |
| 1-9,12     | 820474      | FP ADULT ADJ 90D 16/17 R          | RIGHT   |
| 1-9,12     | 820475      | FP ADULT ADJ 90D 16/17 L          | LEFT    |
| 1-9,12     | 820476      | FP ADULT ADJ 90D 18-20 R          | RIGHT   |
| 1-9,12     | 820477      | FP ADULT ADJ 90D 18-20 L          | LEFT    |
| 1-9,12     | 921474      | ADULT ANGL ADJ 90D 10/11 OMIT H/L | PAIR    |
| 1-9,12     | 921476      | ADULT ANGL ADJ 90D 12/13 OMIT H/L | PAIR    |
| 1-9,12     | 921330      | ADULT ANGL ADJ 90D 14/15 OMIT H/L | PAIR    |
| 1-9,12     | 921331      | ADULT ANGL ADJ 90D 16/17 OMIT H/L | PAIR    |
| 1-9,12     | 921332      | ADULT ANGL ADJ 90D 18-20 OMIT H/L | PAIR    |
| 1-12,14,15 | 820540      | F/PLT ADULT ADJ 90D 10/11 R W/ HL | RIGHT   |
| 1-12,14,15 | 820550      | F/PLT ADULT ADJ 90D 10/11 L W/ HL | LEFT    |
| 1-12,14,15 | 820542      | F/PLT ADULT ADJ 90D 12/13 R W/ HL | RIGHT   |
| 1-12,14,15 | 820552      | F/PLT ADULT ADJ 90D 12/13 L W/ HL | LEFT    |
| 1-12,14,15 | 820478      | F/PLT ADULT ADJ 90D 14/15 R W/ HL | RIGHT   |
| 1-12,14,15 | 820479      | F/PLT ADULT ADJ 90D 14/15 L W/ HL | LEFT    |
| 1-12,14,15 | 820480      | F/PLT ADULT ADJ 90D 16/17 R W/ HL | RIGHT   |
| 1-12,14,15 | 820481      | F/PLT ADULT ADJ 90D 16/17 L W/ HL | LEFT    |
| 1-12,14,15 | 820482      | F/PLT ADULT ADJ 90D 18-20 R W/ HL | RIGHT   |
| 1-12,14,15 | 820483      | F/PLT ADULT ADJ 90D 18-20 L W/ HL | LEFT    |
| 1-12,14,15 | 921470      | F/PLT ADULT ADJ 90D 10/11 W/ HL   | PAIR    |
| 1-12,14,15 | 921472      | F/PLT ADULT ADJ 90D 12/13 W/ HL   | PAIR    |
| 1-12,14,15 | 921333      | F/PLT ADULT ADJ 90D 14/15 W/ HL   | PAIR    |
| 1-12,14,15 | 921334      | F/PLT ADULT ADJ 90D 16/17 W/ HL   | PAIR    |
| 1-12,14,15 | 921335      | F/PLT ADULT ADJ 90D 18-20 W/ HL   | PAIR    |

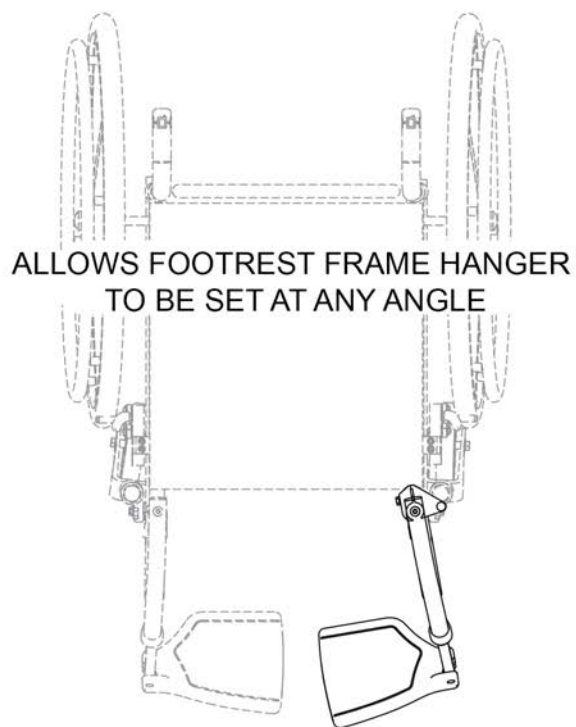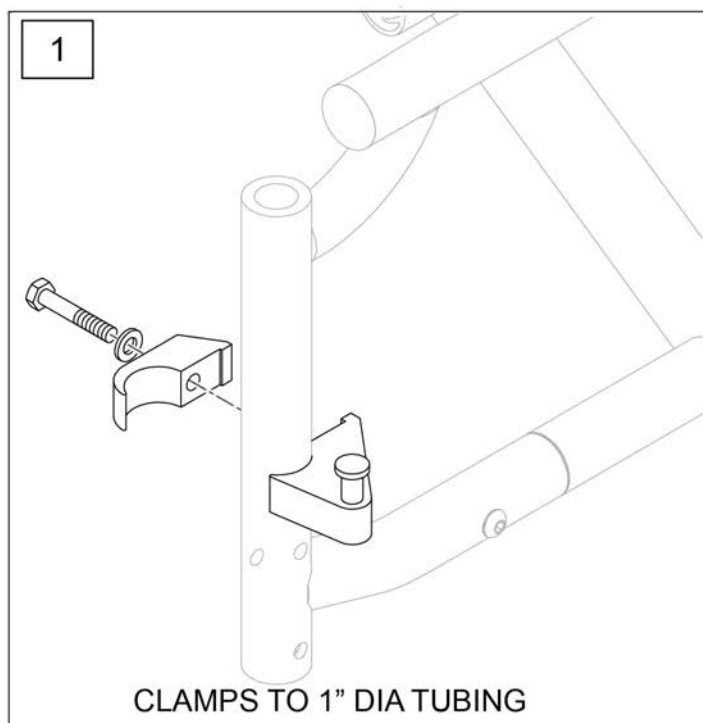

| Pos. | Item Number | Description                    | Remarks |
|------|-------------|--------------------------------|---------|
| 1    | 112807      | LATCH STUD SUPPORT R MULTIPOS  | Single  |
| 1    | 112808      | LATCH STUD SUPPORT L MULTIPOS  | Single  |
| 1    | 112809      | LATCH STUD SUPPORT PR MULTIPOS | Pair    |


**NOTE:** FOR ARMRESTS PRIOR TO 7/22/96 USE RETRO FIT KIT TO REPLACE ADJUSTABLE FLIP-UP ARM CLAMP.

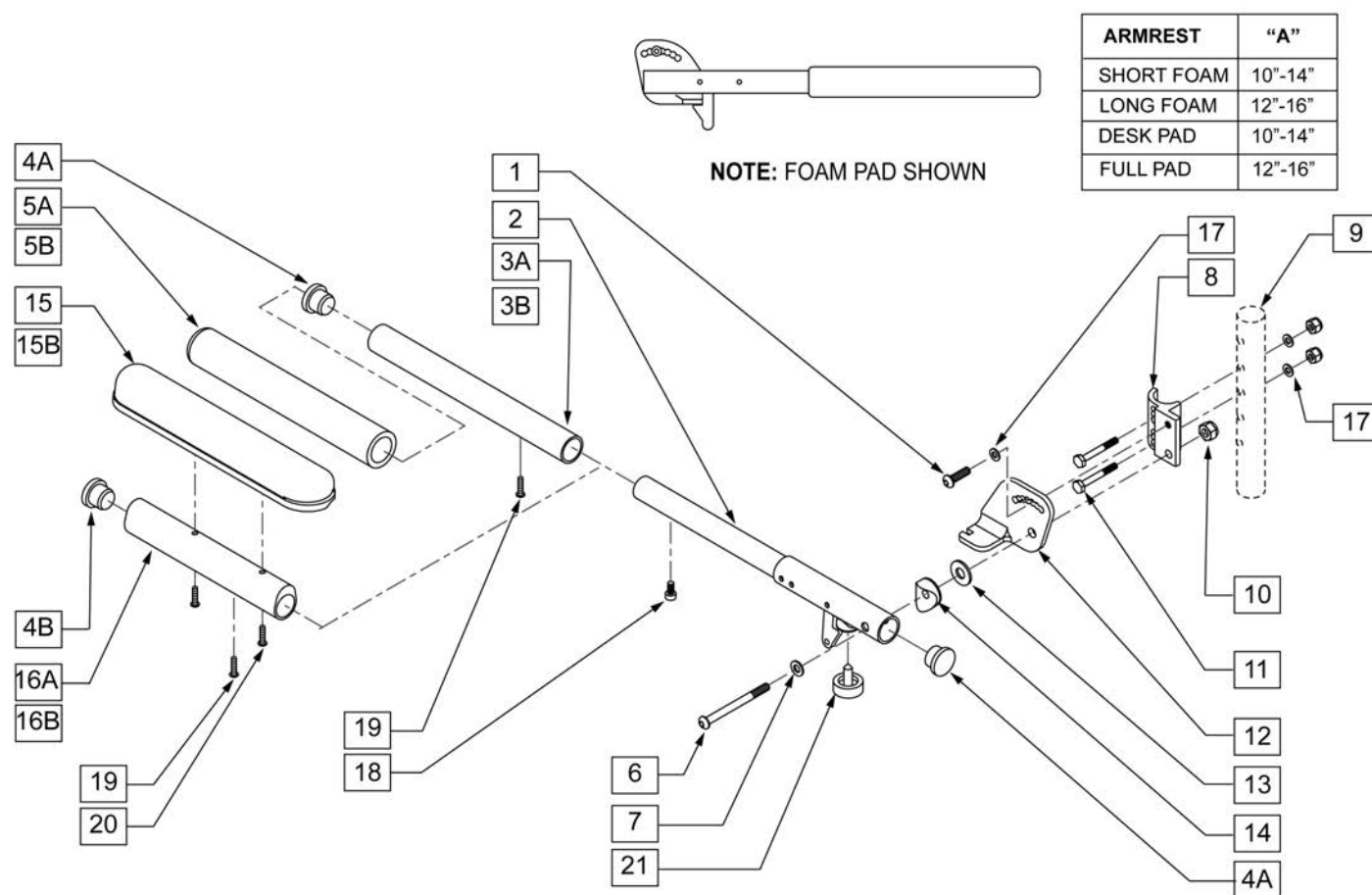

| Pos. | Item Number | Description                       | Remarks   |
|------|-------------|-----------------------------------|-----------|
| 1    | 107025      | BHCS 1/4-28 x 1/2 ROHS            |           |
| 4A   | 561002      | 1" INSERT PLUG .083               |           |
| 4B   | 561062      | 1" INSERT PLUG .065               |           |
| 5A   | 560299      | 7 x .940 x .20 FOAM GRIP          | SHORT PAD |
| 5B   | 560640      | 9" X .93" X .155" FOAM GRIP       | LONG PAD  |
| 6    | 106983      | BHCS 5/16-18 x 2 SHLDR ROHS       |           |
| 7    | 106947      | WASHER FLAT THIN AN 5/16 ROHS     |           |
| 8    | 550168      | ADJUSTABLE LOCK FLIP-UP ARM MOUNT | RIGHT     |
| 8    | 550169      | ADJUSTABLE LOCK FLIP-UP ARM MOUNT | LEFT      |
| 9    | 106935      | NYLOCK NUT THIN 1/4 -28 ROHS      |           |
| 10   | 202117      | 5/16-18 NYLOCK NUT THICK BLK      |           |
| 11   | 106972      | HHCS 1/4-28 x 1 1/2 ROHS          |           |
| 12   | 550474      | ADJUSTABLE FLIP-BACK ARM BRACKET  | RIGHT     |
| 12   | 550475      | ADJUSTABLE FLIP-BACK ARM BRACKET  | LEFT      |
| 13   | 561016      | 1" WASHER 5/16" HOLE              |           |
| 14   | 561006      | 1" SADDLE 5/16" HOLE              |           |
| 15A  | 561024      | ARMREST PAD 10" DESK              |           |

| Pos.                        | Item Number | Description                           | Remarks                     |
|-----------------------------|-------------|---------------------------------------|-----------------------------|
| 15B                         | 561031      | ARMREST PAD 14" FULL                  |                             |
| 17                          | 106943      | WASHER FLAT AN 1/4 ROHS               |                             |
| 18                          | 107036      | SHCS 8 -32 x 3/8 ROHS                 |                             |
| 19                          | 225110      | 10-32 X 1/2" SET SCREW                |                             |
| 20                          | 107022      | BHCS 10-32 X 5/8 ROHS                 |                             |
| 21                          | 560252      | 1" COVED STEM BUMPER                  |                             |
| 3B, 4A, 5B, 19              | 882074      | FOAM PAD LONG ADJUSTABLE LENGTH       | LONG PAD ASSM ONLY          |
| 3A, 4A, 5A, 19              | 882075      | FOAM PAD LONG ADJUSTABLE LENGTH       | SHORT PAD ASSM ONLY         |
| 4B,15B,16B,19,20            | 882070      | FULL PAD ADJUSTABLE LENGTH ASSM       | RIGHT FULL PAD ASSM ONLY    |
| 4B,15B,16B,19,20            | 882071      | FULL PAD ADJUSTABLE LENGTH ASSM       | LEFT FULL PAD ASSM ONLY     |
| 4B,15A,16A,19,20            | 882072      | DESK PAD ADJUSTABLE LENGTH ASSM       | RIGHT DESK PAD ASSM ONLY    |
| 4B,15A,16A,19,20            | 882073      | DESK PAD ADJUSTABLE LENGTH ASSM       | LEFT DESK PAD ASSM ONLY     |
| 1,2,6-14,17,22              | 882088      | ADJ LENGTH LOCKING FLIP-UP ASSM R     | RIGHT ARM ASSM W/O PAD      |
| 1,2,6-14,17,22              | 882089      | ADJ LENGTH LOCKING FLIP-UP ASSM R     | LEFT ARM ASSM W/O PAD       |
| 1,2,3B,4A,5B,6-14,17,19 -21 | 900454      | ADJ LENGTH LOCKING FLIP-UP FOAM LONG  | PAIR - LONG FOAM ARM ASSM   |
| 1,2,3B,4A,5B,6-14,17,19 -21 | 881294      | ADJ LENGTH LOCKING FLIP-UP FOAM LONG  | RIGHT - LONG FOAM ARM ASSM  |
| 1,2,3B,4A,5B,6-14,17,19 -21 | 881295      | ADJ LENGTH LOCKING FLIP-UP FOAM LONG  | LEFT - LONG FOAM ARM ASSM   |
| 1,2,3A,4A,5A,6-14,17,19 -21 | 900455      | ADJ LENGTH LOCKING FLIP-UP FOAM SHORT | PAIR - SHORT FOAM ARM ASSM  |
| 1,2,3A,4A,5A,6-14,17,19 -21 | 881296      | ADJ LENGTH LOCKING FLIP-UP FOAM SHORT | RIGHT - SHORT FOAM ARM ASSM |
| 1,2,3A,4A,5A,6-14,17,19 -21 | 881297      | ADJ LENGTH LOCKING FLIP-UP FOAM SHORT | LEFT - SHORT FOAM ARM ASSM  |
| 1,2,4B,6-14,15B,16B,17-21   | 900452      | ADJ LENGTH LOCKING FLIP-UP ASSM FULL  | PAIR - FULL PAD ARM ASSM    |
| 1,2,4B,6-14,15B,16B,17-21   | 881290      | ADJ LENGTH LOCKING FLIP-UP FULL PAD   | RIGHT - FULL PAD ARM ASSM   |
| 1,2,4B,6-14,15B,16B,17-21   | 881291      | ADJ LENGTH LOCKING FLIP-UP FULL PAD   | LEFT - FULL PAD ARM ASSM    |
| 1,2,4B,6-14,15A,16A,17-21   | 900453      | ADJ LENGTH LOCKING FLIP-UP ASSM DESK  | PAIR - DESK PAD ARM ASSM    |
| 1,2,4B,6-14,15A,16A,17-21   | 881292      | ADJ LENGTH LOCKING FLIP-UP DESK PAD   | RIGHT - DESK PAD ARM ASSM   |
| 1,2,4B,6-14,15A,16A,17-21   | 881293      | ADJ LENGTH LOCKING FLIP-UP DESK PAD   | LEFT - DESK PAD ARM ASSM    |


**NOTE:** If ordering for pediatric chairs with 20" or 22" wheels, you must order left armrest pad for right side, and right armrest pad for left side.

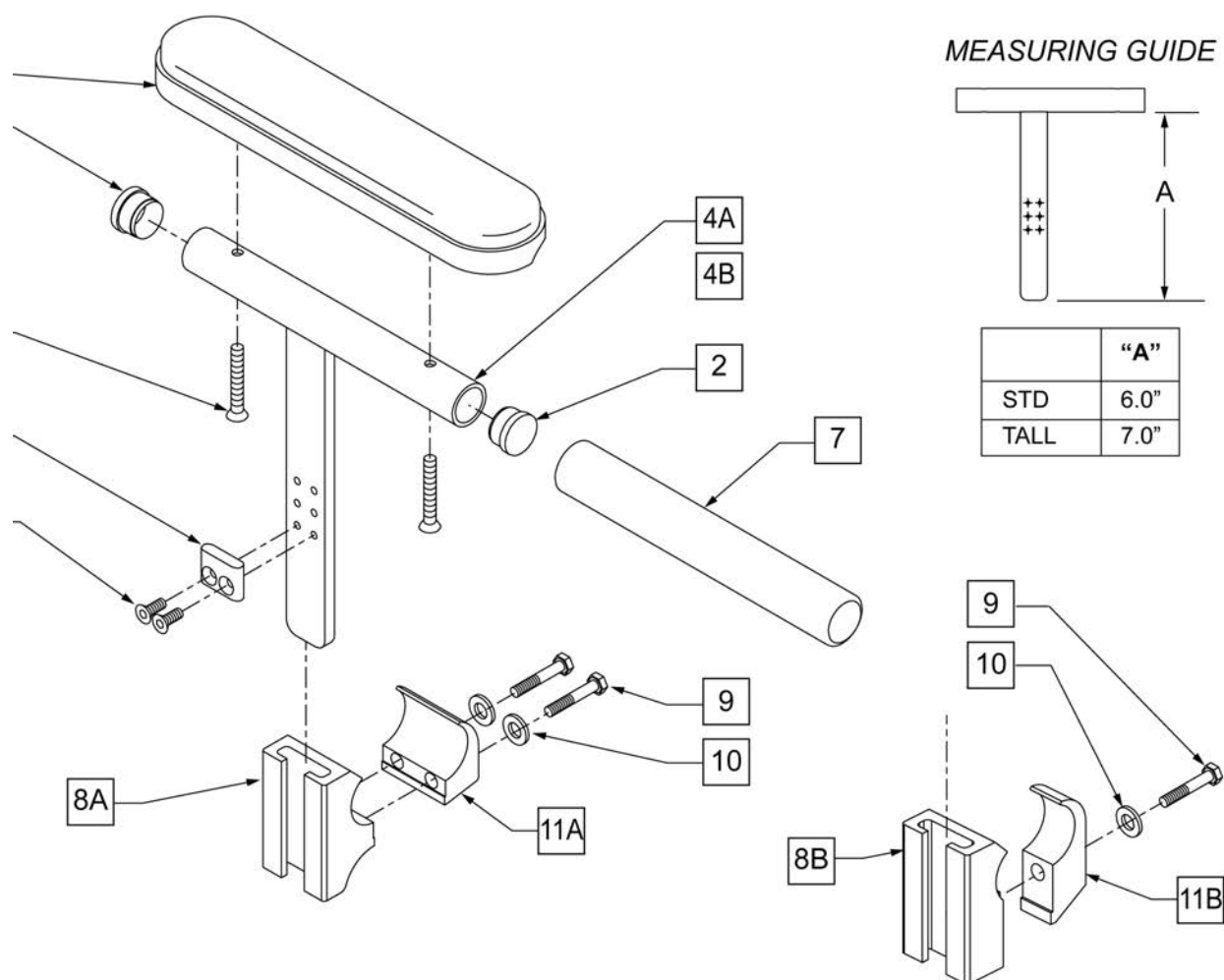


| Pos.       | Item Number | Description                  | Remarks                            |
|------------|-------------|------------------------------|------------------------------------|
| 1          | 560780      | ARMREST PAD SMALL 9"         |                                    |
| 2          | 561007      | 7/8" INSERT PLUG .083        |                                    |
| 3          | 107023      | 10-32 x 1 3/8 BH C/S BLK     |                                    |
| 5          | 550105-QU   | KIDZ ARM STOP                |                                    |
| 6          | 113279      | 10-32 x 1/2 FH C/S BLK       |                                    |
| 7          | 560309      | ARMREST FOAM KIDZ            |                                    |
| 8A         | 502037      | TPHHA & SG CLAMP 1"          | USED W/ ZIPPIE TS AND ZIPPIE GS    |
| 8B         | 550325      | SIDE GUARD/KIDZ ARM REC      | USED WITH ZIPPIE IRIS AND ZIPPIE 2 |
| 9          | 106971      | 1/4-28 x 1 3/8 HEX C/S       |                                    |
| 10         | 106944      | 1/4 AN WASHER FLAT THIN      |                                    |
| 11A        | 502038      | TPHAR & SGR CLAMP 1"         | USED W/ ZIPPIE TS AND ZIPPIE GS    |
| 11B        | 550251      | CLAMP P300 CHRNGN/ADJ FL KDS | USED WITH ZIPPIE IRIS AND ZIPPIE 2 |
| 1-3,4A,5,6 | 880162      | KIDS ARM STANDARD R          | RIGHT                              |
| 1-3,4A,5,6 | 880163      | KIDS ARM STANDARD L          | LEFT                               |

| Pos.                   | Item Number | Description                      | Remarks                                    |
|------------------------|-------------|----------------------------------|--------------------------------------------|
| 1-3,4B,5,6             | 880164      | KIDZ ARM TALL R                  | RIGHT                                      |
| 1-3,4B,5,6             | 880165      | KIDZ ARM TALL L                  | LEFT                                       |
| 8A,9,10,11A            | 882681      | T-PST/SDE GUARD REC W/CLMP ZTS   | INCLUDES 1 RECEIVER -FITS ZIPPPIE TS & GS  |
| 8B,9,10,11B            | 881200      | SIDE GUARD/KIDZ ARM REC W/ CLAMP | INCLUDES 1 RECEIVER -FITS ZIPPPIE IRIS & 2 |
| 2,4A,5-7               | 885010      | KIDZ ARM STD W/FOAM R            | RIGHT                                      |
| 2,4A,5-7               | 885011      | KIDZ ARM STD W/FOAM L            | LEFT                                       |
| 2,4B,5-7               | 885012      | KIDZ ARM TALL W/FOAM R           | RIGHT                                      |
| 2,4B,5-7               | 885013      | KIDZ ARM TALL W/FOAM L           | LEFT                                       |
| 1-3,4A,5,6,8A,9,10,11A | 900795      | KIDZ ARM ASSM STD W/REC ZTS      | PAIR -FITS ZIPPPIE TS & GS                 |
| 1-3,4B,5,6,8A,9,10,11A | 900796      | KIDZ ARM ASSM TALL W/REC ZTS     | PAIR -FITS ZIPPPIE TS & GS                 |
| 2,4A,5-7,8A,9,10,11A   | 900797      | KDZ ARM ASSM STD W/FM W/RC ZTS   | PAIR -FITS ZIPPPIE TS & GS                 |
| 2,4B,5-7,8A,9,10,11A   | 900798      | KDZ ARM ASSM TLL W/FM W/RC ZTS   | PAIR -FITS ZIPPPIE TS & GS                 |
| 1-3,4A,5,6,8B,9,10,11B | 900693      | KIDZ ARM ASSM STD W/REC          | PAIR -FITS ZIPPPIE IRIS & 2                |
| 1-3,4B,5,6,8B,9,10,11B | 900694      | KIDZ ARM ASSM TALL W/REC         | PAIR -FITS ZIPPPIE IRIS & 2                |
| 2,4A,5-7,8B,9,10,11B   | 900755      | KIDZ ARM ASSM STD W/FOAM W/REC   | PAIR -FITS ZIPPPIE IRIS & 2                |
| 2,4B,5-7,8B,9,10,11B   | 900756      | KIDZ ARM ASSM TALL W/FOAM W/REC  | PAIR -FITS ZIPPPIE IRIS & 2                |



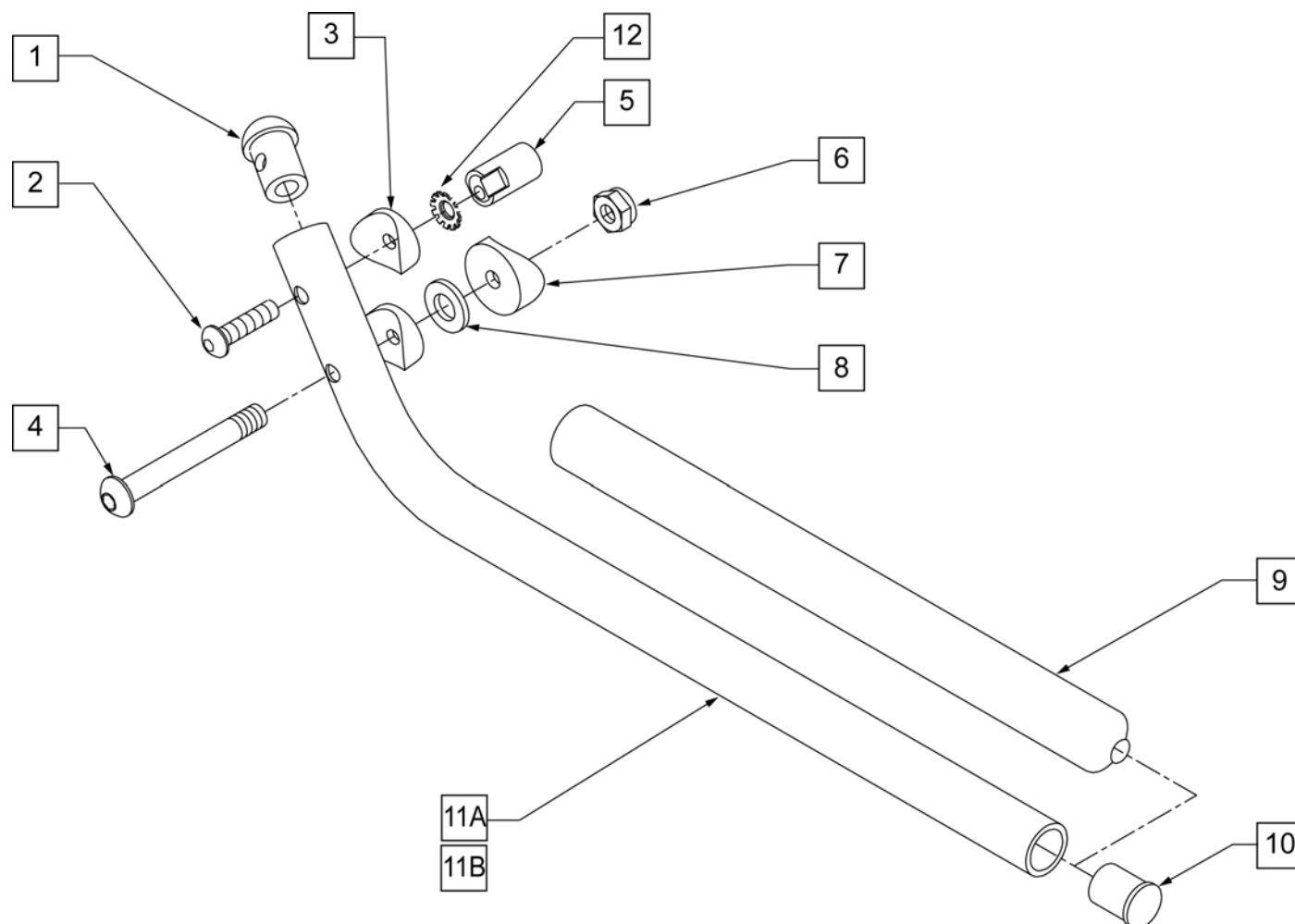


| Pos.     | Item Number | Description                                   | Remarks |
|----------|-------------|-----------------------------------------------|---------|
| 1        | 561034      | SHORT DOME PLUG                               |         |
| 2        | 107028      | 1/4-28 x 1 1/2 BH C/S                         |         |
| 3        | 561003      | 3/4" SADDLE THIN                              |         |
| 4        | 106966      | 1/4-28 x 2 1/4 BH C/S                         |         |
| 5        | 483041      | FLIP-UP ARM CAM ZP/P5                         |         |
| 6        | 106935      | 1/4-28 NYLOCK NUT THIN                        |         |
| 7        | 561004      | 1" SADDLE 1/4" HOLE                           |         |
| 8        | 106944      | 1/4 AN WASHER FLAT THIN                       |         |
| 9        | 560599      | 8 x .705 x .200 FOAM GRIP                     |         |
| 10       | 561000-QU   | 3/4" INSERT PLUG .083                         |         |
| 12       | 267350      | 1/4" WASHER INT LOCK ZN                       |         |
| 1-11A,12 | 880268      | ZP FLIP-UP ARM ASSEMBLY SEAT DEPTH 10"-13"    | RIGHT   |
| 1-11A,12 | 880269      | ZP FLIP-UP ARM ASSEMBLY SEAT DEPTH 10"-13"    | LEFT    |
| 1-11A,12 | 900446      | FLIP-UP ARM ASSEMBLY 10"-13" ZP               | PAIR    |
| 1-11B,12 | 880270      | FLIP-UP ARM ASSEMBLY SEAT DEPTH 14"-16" ZP/P5 | RIGHT   |
| 1-11B,12 | 880271      | FLIP-UP ARM ASSEMBLY SEAT DEPTH 14"-16" ZP/P5 | LEFT    |
| 1-11B,12 | 900448      | FLIP-UP ARM ASSEMBLY                          | PAIR    |

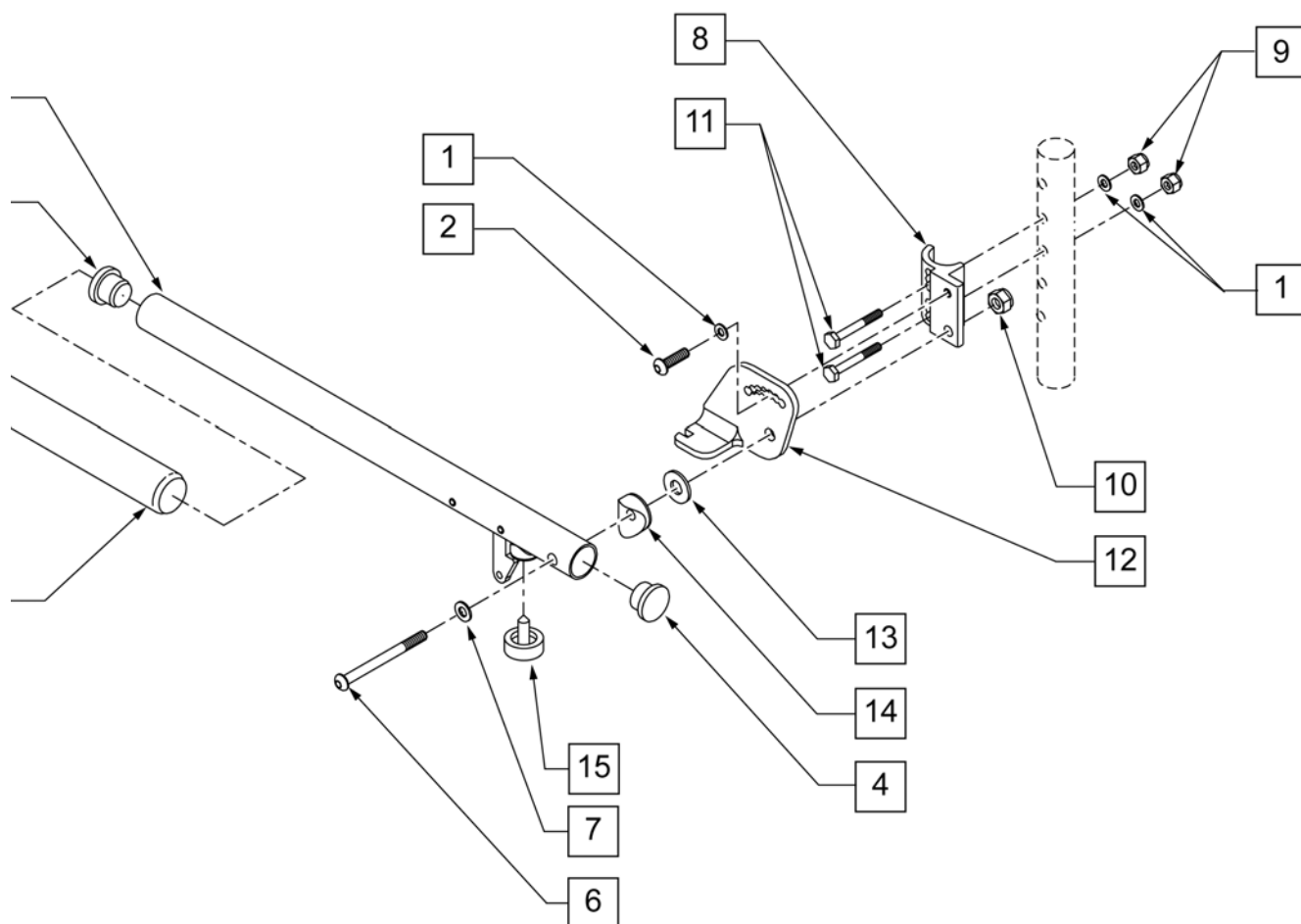


| Pos.       | Item Number | Description                   | Remarks                          |
|------------|-------------|-------------------------------|----------------------------------|
| 1          | 106943      | 1/4 AN WASHER FLAT            |                                  |
| 2          | 107025      | 1/4-28 x 1/2 BH C/S           |                                  |
| 4          | 561002      | 1" INSERT PLUG .083           |                                  |
| 5          | 560640      | 9 x .93 x .155 FOAM GRIP      |                                  |
| 6          | 106983      | 5/16-18 x 2 BH C/S SHT THREAD |                                  |
| 7          | 106947      | 5/16 WASHER FLAT THIN         |                                  |
| 8          | 550168      | ADJ LOCK FLIP-UP ARM MNT R    |                                  |
| 8          | 550169      | ADJ LOCK FLIP-UP ARM MNT L    |                                  |
| 9          | 106935      | 1/4-28 NYLOCK NUT THIN        |                                  |
| 10         | 202117      | 5/16-18 NYLOCK NUT THICK BLK  |                                  |
| 11         | 106972      | 1/4-28 x 1 1/2 G8 HEX C/S     |                                  |
| 12         | 550474      | ADJ FLIP-BACK ARM BRACKET R   |                                  |
| 12         | 550475      | ADJ FLIP BACK ARM BRACKET L   |                                  |
| 13         | 561016      | 1" WASHER 5/16" HOLE          |                                  |
| 14         | 561006      | 1" SADDLE 5/16" HOLE          |                                  |
| 15         | 560252      | 1" COVED STEM BUMPER          |                                  |
| 3-5        | 880274      | ADJ FLIP-BACK TUBE W/LATCH    |                                  |
| 1,2,6,8-10 | 881272      | ADJ LOCK F-UP ARMREST RETRO R | USE FOR ARMREST PRIOR TO 7/22/96 |

| Pos.       | Item Number | Description                   | Remarks                         |
|------------|-------------|-------------------------------|---------------------------------|
| 1,2,6,8-10 | 881273      | ADJ LOCK F-UP ARMREST RETRO L | USE FOR ARMRST PRIOR TO 7/22/96 |
| 1-15       | 880272      | ADJ FLIP BACK ARMREST R       |                                 |
| 1-15       | 880273      | ADJ FLIP BACK ARMREST L       |                                 |
| 1-15       | 900449      | ADJ LOCK FLIP-UP ARMREST ASSM | PAIR                            |

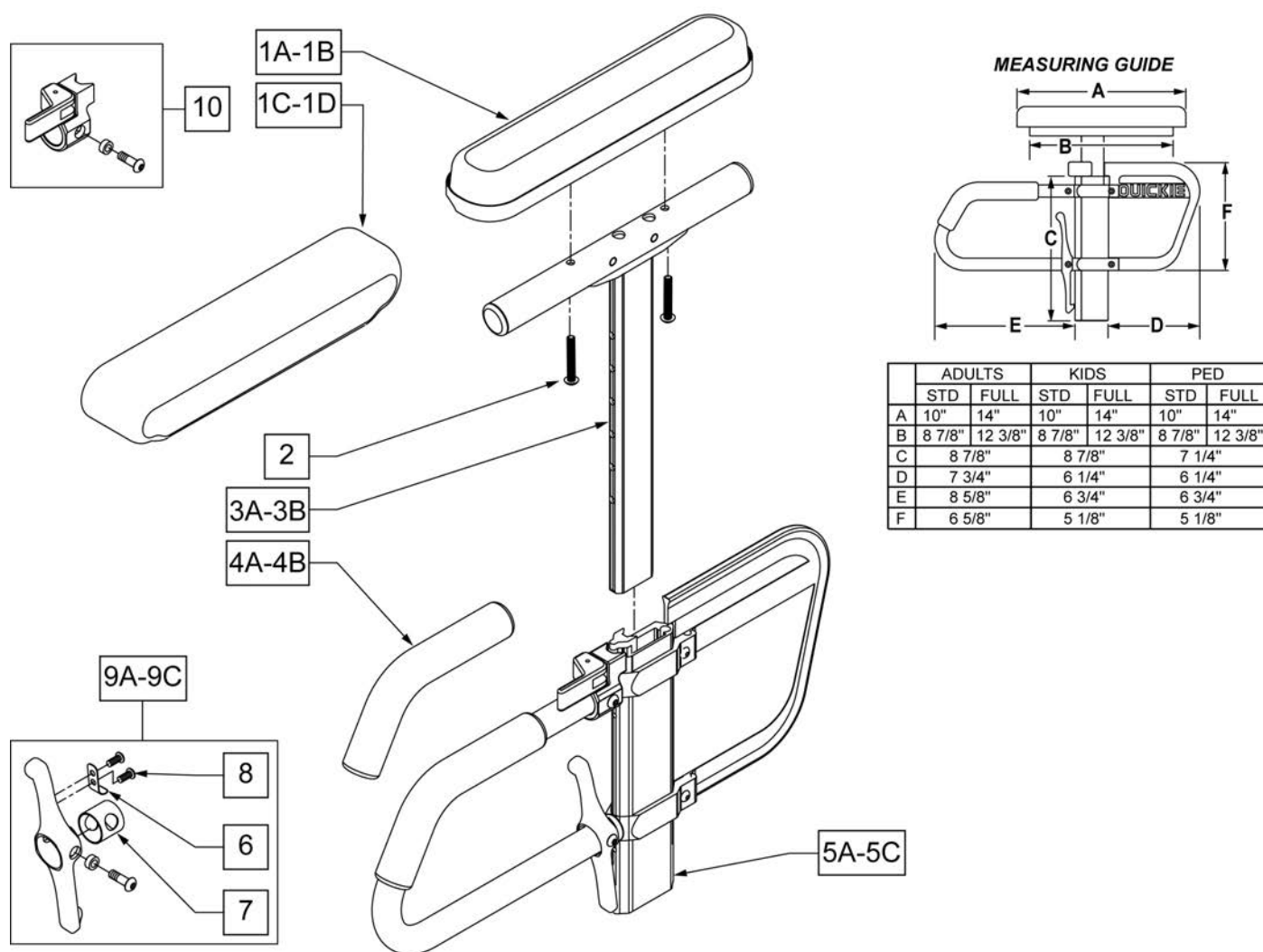

| Pos. | Item Number | Description                    | Remarks                    |
|------|-------------|--------------------------------|----------------------------|
| 1A   | 561024      | ARMREST PAD 10" DESK           | DESK PAD                   |
| 1B   | 561031      | ARMREST PAD 14" FULL           | FULL PAD                   |
| 1C   | 561280      | ARMREST PAD 10" L W/F          | LEFT - DESK WATERFALL PAD  |
| 1C   | 561281      | ARMREST PAD 10" R W/F          | RIGHT - DESK WATERFALL PAD |
| 1D   | 561284      | ARMREST PAD 14" L W/F          | LEFT - FULL WATERFALL PAD  |
| 1D   | 561285      | ARMREST PAD 14" R W/F          | RIGHT - FULL WATERFALL PAD |
| 2    | 226038      | 10-32 x 1 3/8 BH C/S BLK       |                            |
| 4A   | 560610      | 5" x .705 x .125 FOAM TUBE     | ADULT                      |
| 4B   | 560627      | 7-1/2" x .705 x .125 FOAM TUBE | PEDS/KIDS                  |
| 5A   | 882084      | HT ADJ LOWER DC ARM ASSM STD R | RIGHT - STANDARD/ADULT     |
| 5A   | 882085      | HT ADJ LOWER DC ARM ASSM STD L | LEFT - STANDARD/ADULT      |
| 5B   | 882082      | HT ADJ LOWER DC ARM ASM KIDS R | RIGHT - KIDS               |
| 5B   | 882083      | HT ADJ LOWER DC ARM ASM KIDS L | LEFT - KIDS                |
| 5C   | 882080      | HT ADJ LOWER ARM ASSM PE D R   | RIGHT - PEDIATRIC          |
| 5C   | 882081      | HT ADJ LOWER ARM ASSM PE D L   | LEFT - PEDIATRIC           |
| 6    | 321721      | LEVER RETURN SPRING            |                            |
| 7    | 562825      | RELEASE LEVER INNER SLEEVE     |                            |
| 8    | 281045      | #6 x 1/4 PAN PHIL S/M/S        |                            |

| Pos.     | Item Number | Description                         | Remarks                              |
|----------|-------------|-------------------------------------|--------------------------------------|
| 9A       | 888000      | HT ADJ REC REL LEVER ASSM STD       | ADULT                                |
| 9B       | 888001      | HT ADJ REC REL LEVER ASSM KIDS      | KIDS                                 |
| 9C       | 888002      | HT ADJ REC REL LEVER ASSM PEDS      | PEDS                                 |
| 10       | 888003      | HT ADJ UPPER ARM REL ASSM R         | RIGHT                                |
| 10       | 888004      | HT ADJ UPPER ARM REL ASSM L         | LEFT                                 |
| >>>>     |             | ADULT UPPER ASSEMBLIES              |                                      |
| 1A-3A    | 880192      | HT ADJ UPPER ARM ASSM DESK R        | RIGHT - ADULT DESK PAD               |
| 1A-3A    | 880198      | HT ADJ UPPER ARM ASSM DESK L        | LEFT - ADULT DESK PAD                |
| 1B-3A    | 880193      | HT ADJ UPPER ARM ASSM FL R          | RIGHT - ADULT FULL PAD               |
| 1B-3A    | 880199      | HT ADJ UPPER ARM ASSM FL L          | LEFT - ADULT FULL PAD                |
| >>>>     |             | ADULT UPPER ASSEMBLIES              |                                      |
| 1C-3A    | 881230      | HT ADJ UPPER L ARM ASSM DESK W/F    | RIGHT - ADULT DESK WATER FALL PAD    |
| 1C-3A    | 881231      | HT ADJ UPPER R ARM ASSM DESK W/F    | LEFT - ADULT DESK WATER FALL PAD     |
| 1D-3A    | 881232      | HT ADJ UPPER L ARM ASSM FL W/F      | RIGHT - ADULT FULL WATER FALL PAD    |
| 1D-3A    | 881233      | HT ADJ UPPER R ARM ASSM FL W/F      | LEFT - ADULT FULL WATER FALL PAD     |
| >>>>     |             | PEDIATRIC AND KIDS UPPER ASSEMBLIES |                                      |
| 1A-3B    | 880174      | HT ADJ UPPER ARM PED PWR DESK R     | RIGHT - PED/KIDS DESK PAD            |
| 1A-3B    | 880175      | HT ADJ UPPER ARM PED PWR DESK L     | LEFT - PED/KIDS DESK PAD             |
| 1B-3B    | 880176      | HT ADJ UPPER ARM PED PWR FULL R     | RIGHT - PED/KIDS FULL PAD            |
| 1B-3B    | 880177      | HT ADJ UPPER ARM PED PWR FULL L     | LEFT - PED/KIDS FULL PAD             |
| >>>>     |             | PEDIATRIC AND KIDS UPPER ASSEMBLIES |                                      |
| 1C-3B    | 888341      | HT ADJ PED UPPER R ARM ASSM D WF    | RIGHT - PED/KIDS DESK WATER FALL PAD |
| 1C-3B    | 888342      | HT ADJ PED UPPER L ARM ASSM D WF    | LEFT - PED/KIDS DESK WATER FALL PAD  |
| 1D-3B    | 888343      | HT ADJ PED UPPER R ARM ASSM F WF    | RIGHT - PED/KIDS FULL WATER FALL PAD |
| 1D-3B    | 888344      | HT ADJ PED UPPER L ARM ASSM F WF    | LEFT - PED/KIDS FULL WATER FALL PAD  |
| >>>>     |             | COMPLETE ADULT ARMREST ASSEMBLY     |                                      |
| 1A-3A-5A | 880210      | HT ADJ STD R W/DESK ARM PAD         | RIGHT - ADULT - DESK PAD             |
| 1A-3A-5A | 880211      | HT ADJ STD L W/DESK ARM PAD         | LEFT - ADULT - DESK PAD              |
| 1B-3A-5A | 880212      | HT ADJ STD R W/FULL ARM PAD         | RIGHT - ADULT - FULL PAD             |
| 1B-3A-5A | 880213      | HT ADJ STD L W/FULL ARM PAD         | LEFT - ADULT - FULL PAD              |
| 1C-3A-5A | 881301      | HT ADJ STD R W/DESK W/F PAD         | RIGHT - ADULT - WATERFALL DESK PAD   |
| 1C-3A-5A | 881300      | HT ADJ STD L W/DESK W/F PAD         | LEFT - ADULT - WATERFALL DESK PAD    |
| 1D-3A-5A | 881305      | HT ADJ STD R W/FULL W/F PAD         | RIGHT - ADULT - WATERFALL FULL PAD   |
| 1D-3A-5A | 881304      | HT ADJ STD L W/FULL W/F PAD         | LEFT - ADULT - WATERFALL FULL PAD    |
| >>>>     |             | COMPLETE KIDS ARMREST ASSEMBLY      |                                      |
| 1A-3B-5B | 880214      | HT ADJ KIDS R W/DESK ARM PAD        | RIGHT - KIDS - DESK PAD              |
| 1A-3B-5B | 880215      | HT ADJ KIDS L W/DESK ARM PAD        | LEFT - KIDS - DESK PAD               |
| 1B-3B-5B | 880216      | HT ADJ KIDS R W/FULL ARM PAD        | RIGHT - KIDS - FULL PAD              |
| 1B-3B-5B | 880217      | HT ADJ KIDS L W/FULL ARM PAD        | LEFT - KIDS - FULL PAD               |
| 1C-3B-5B | 881303      | HT ADJ KIDS R W/DESK W/F PAD        | RIGHT - KIDS - WATERFALL DESK PAD    |
| 1C-3B-5B | 881302      | HT ADJ KIDS L W/DESK W/F PAD        | LEFT - KIDS - WATERFALL DESK PAD     |
| 1D-3B-5B | 881307      | HT ADJ KIDS R W/FULL W/F PAD        | RIGHT - KIDS - WATERFALL FULL PAD    |
| 1D-3B-5B | 881306      | HT ADJ KIDS L W/FULL W/F PAD        | LEFT - KIDS - WATERFALL FULL PAD     |
| >>>>     |             | COMPLETE PEDIATRIC ARMREST ASSEMBLY |                                      |
| 1A-3B-5C | 880226      | HT ADJ PED R W/DESK ARM PAD         | RIGHT - PEDS - DESK PAD              |
| 1A-3B-5C | 880227      | HT ADJ PED L W/DESK ARM PAD         | LEFT - PEDS - DESK PAD               |
| 1B-3B-5C | 880228      | HT ADJ PED R W/FULL ARM PAD         | RIGHT - PEDS - FULL PAD              |
| 1B-3B-5C | 880229      | HT ADJ PED L W/FULL ARM PAD         | LEFT - PEDS - FULL PAD               |
| 1C-3B-5C | 888345      | HT ADJ PED R W/DESK W/F PAD         | RIGHT - PEDS - WATERFALL DESK PAD    |
| 1C-3B-5C | 888346      | HT ADJ PED L W/DESK W/F PAD         | LEFT - PEDS - WATERFALL DESK PAD     |
| 1D-3B-5C | 888347      | HT ADJ PED R W/FULL W/F PAD         | RIGHT - PEDS - WATERFALL FULL PAD    |
| 1D-3B-5C | 888348      | HT ADJ PED L W/FULL W/F PAD         | LEFT - PEDS - WATERFALL FULL PAD     |

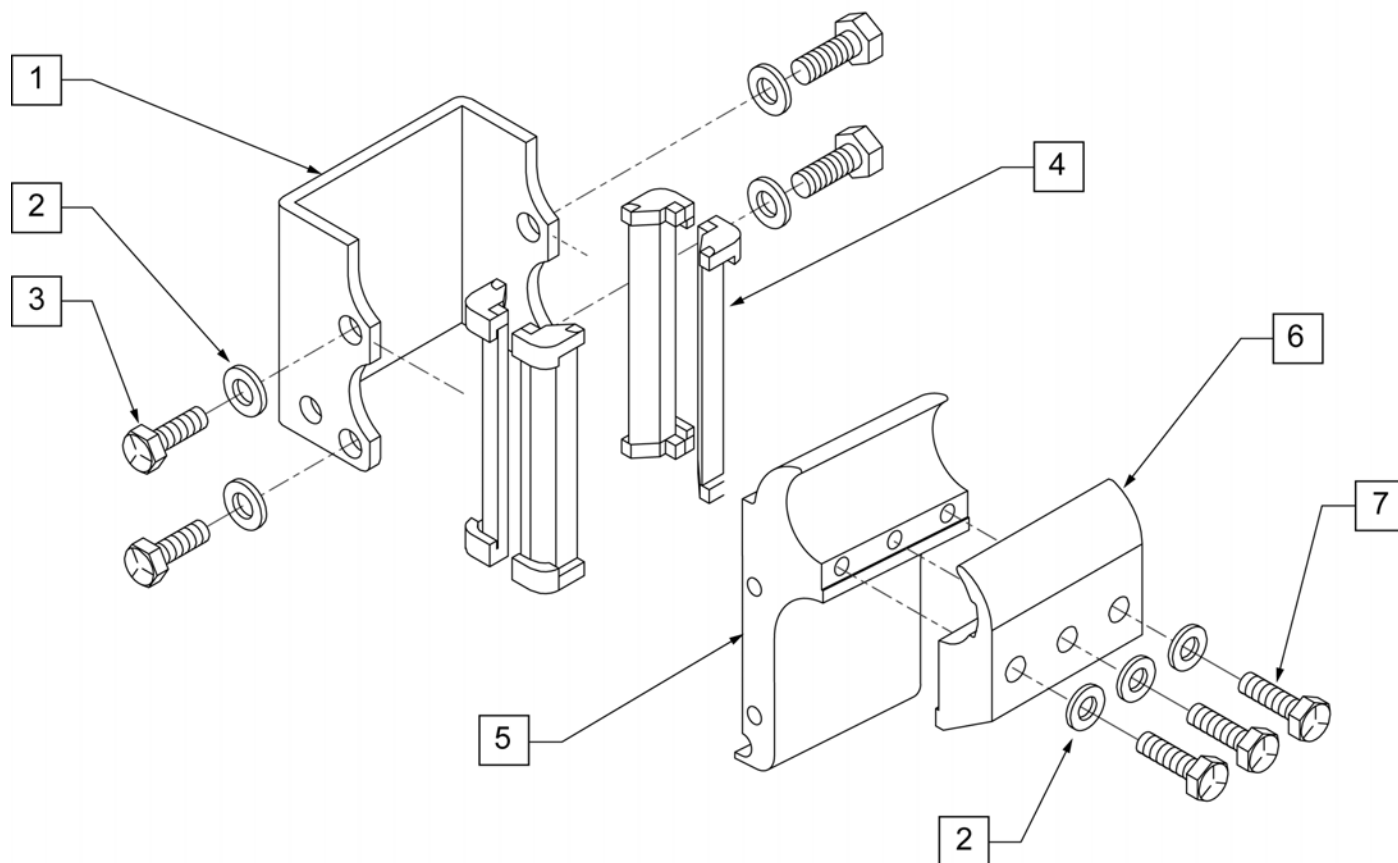

| Pos. | Item Number | Description                                    |
|------|-------------|------------------------------------------------|
| 1    | 550485      | HEIGHT ADJUSTABLE RECEIVER CHANNEL             |
| 2    | 106943      | WASHER FLAT AN 1/4 ROHS                        |
| 3    | 221096      | 1/4-28 x 5/8 HEX C/S                           |
| 4    | 561134      | HEIGHT ADJUSTABLE RECEIVER LINER               |
| 5    | 550528      | HEIGHT ADJUSTABLE RECEIVER ZIPPIE              |
| 6    | 550529      | HEIGHT ADJUSTABLE RECIEVER CLAMP ZIPPIE        |
| 7    | 106970      | HHCS 1/4-28 X 1-1/4 ROHS                       |
| 1 -5 | 882679      | HEIGHT ADJUSTABLE RECIEVER W/O CLAMP ZIPPIE TS |
| 1 -7 | 882680      | HEIGHT ADJUSTABLE RECIEVER W/ CLAMP ZIPPIE TS  |

**NOTE:** USED ON QUICKIE 2 CHAIRS W/ S/N PREFIX Q2..

**NOTE:** USED ON ZIPPIE GS CHAIRS W/ S/N PREFIX ZGS & ZGE.


**NOTE:** USED ON QUICKIE IRIS CHAIRS.

**NOTE:** USED ON QUICKIE LX/LXI CHAIRS.

**NOTE:** USED ON ZIPPIE Z-BOP CHAIRS.


**NOTE:** USED ON ZIPPIE TS CHAIRS

**NOTE:** USED ON BREEZY 600 CHAIRS.

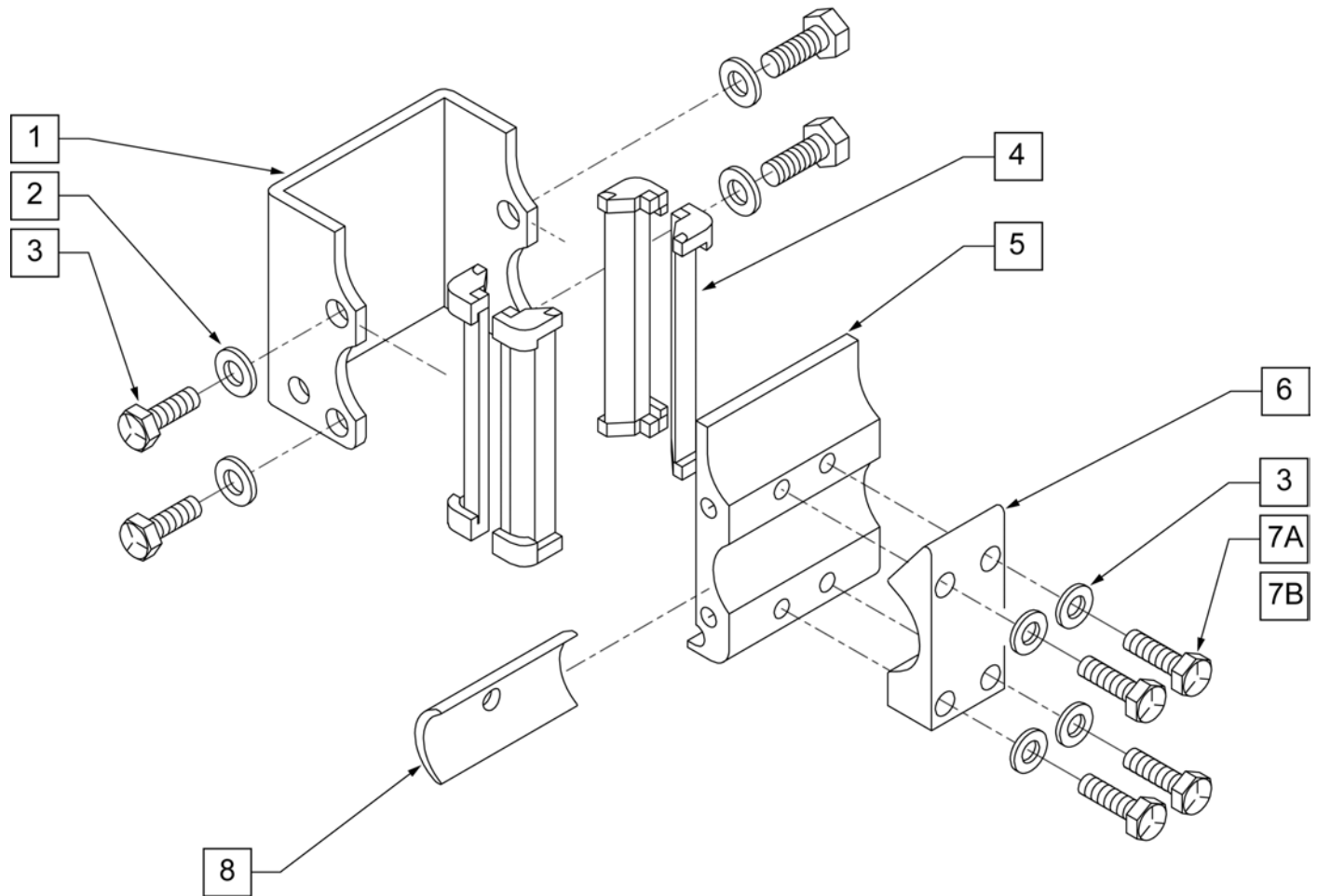


| Pos.   | Item Number | Description                                        | Remarks                                            |
|--------|-------------|----------------------------------------------------|----------------------------------------------------|
| 1      | 550485      | HEIGHT ADJUSTABLE RECEIVER CHANNEL                 |                                                    |
| 2      | 221096      | 1/4-28 x 5/8 HEX C/S                               |                                                    |
| 3      | 106943      | 1/4 AN WASHER FLAT                                 |                                                    |
| 4      | 561134      | HEIGHT ADJUSTABLE RECEIVER LINER                   |                                                    |
| 5      | 550483      | HEIGHT ADJUSTABLE RECEIVER STD                     |                                                    |
| 6      | 550490      | HEIGHT ADJUSTABLE RECEIVER CLAMP 4 HOLE            |                                                    |
| 7A     | 106971      | 1/4-28 x 1 3/8 HEX C/S                             |                                                    |
| 7B     | 106972      | 1/4-28 X 1 1/2 G8 HEX C/S                          | USE WITH RECIEVER SPACER                           |
| 8      | 550486      | HEIGHT ADJUSTABLE RECEIVER SPACER                  | Used to space Armrest 1/8" from frame on each side |
| 1-5    | 880230      | HEIGHT ADJUSTABLE RECIEVER STD W/O CLAMP           |                                                    |
| 1-7A   | 880240      | HEIGHT ADJUSTABLE RECIEVER STD W/ CLAMP            |                                                    |
| 1-7B,8 | 880245      | HEIGHT ADJUSTABLE RECIEVER STD W/ CLAMP AND SPACER |                                                    |




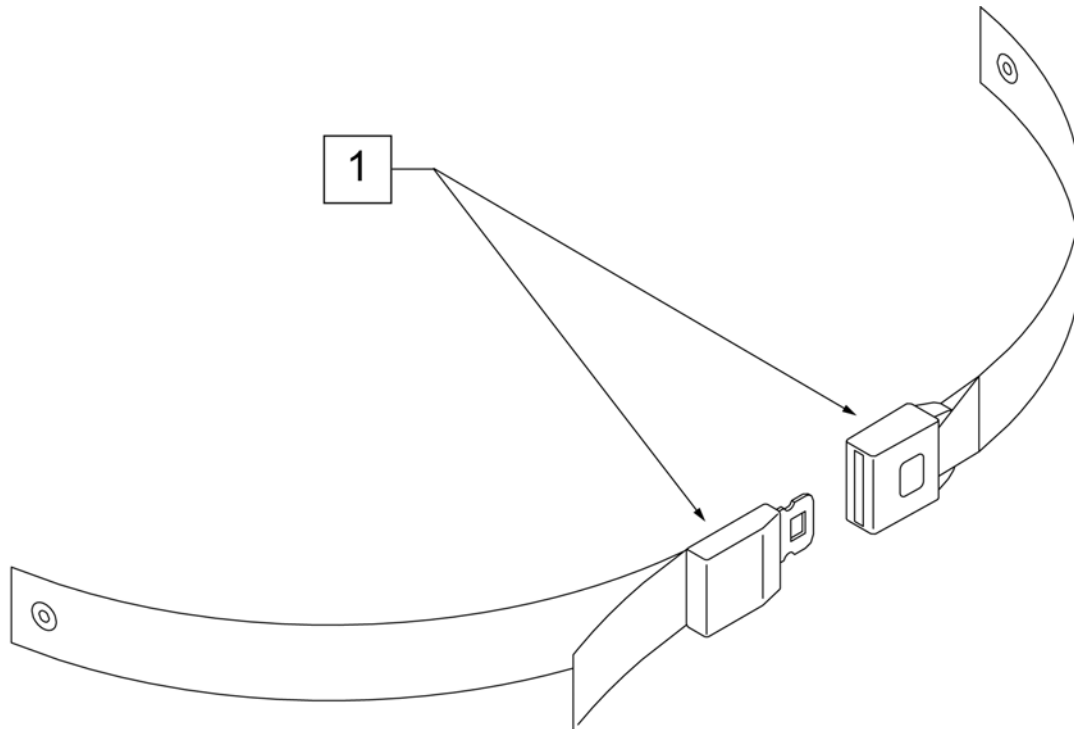


| Pos. | Item Number | Description          | Remarks                           |
|------|-------------|----------------------|-----------------------------------|
| 1    | 750276      | 1 1/2" POSITION BELT | AUTOSTYLE-64" Max length of belt. |

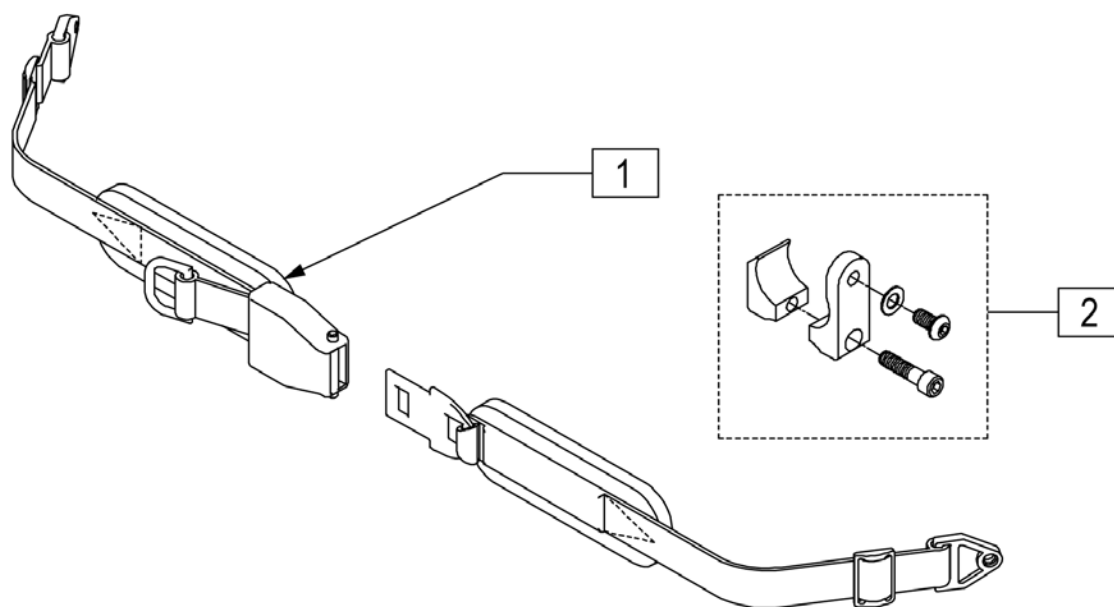

| Pos. | Item Number | Description                      |
|------|-------------|----------------------------------|
| 1    | 450418-QU   | BELT 2" AIRCRAFT BUCKLE          |
| 1    | 450423-QU   | BELT 2" AIRCRAFT BUCKLE W/PAD    |
| 1    | 450424      | BELT 2" AIRCRAFT BKL LONG W/PAD  |
| 1    | 450425      | BELT 2" AIRCRAFT BKL LONG        |
| 1    | 450419-QU   | BELT 2" AIRCRAFT BKL W/QUAD LP   |
| 1    | 468419      | BELT 2" AIRCRT BKL W/QD LPCZ     |
| 2    | 900116      | POSITION BELT ATTACHMENT KIT     |
| 1,2  | 920918      | POS BELT ATT KIT W/2" AIR BKL    |
| 1,2  | 920919      | POS BELT ATT KIT W/2" AIR BKL PD |

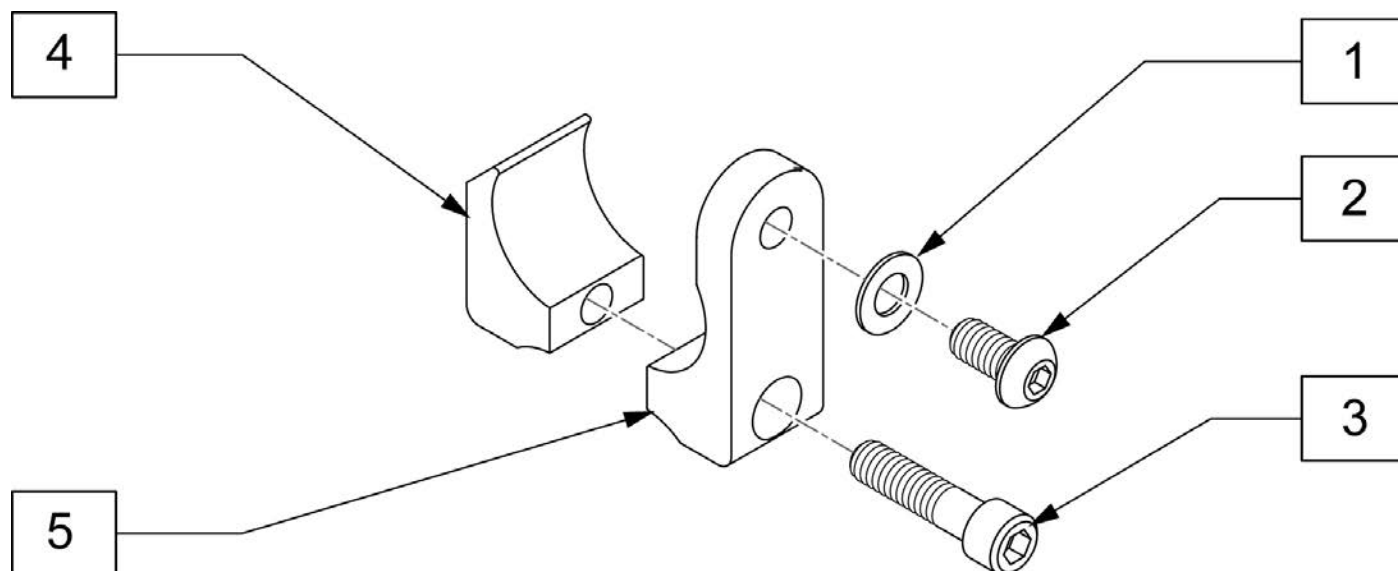


| Pos. | Item Number | Description                  |
|------|-------------|------------------------------|
| 1    | 106943      | WASHER FLAT AN 1/4 ROHS      |
| 2    | 226154      | 1/4-20 X 1/2 BH C/S W/PATCH  |
| 3    | 221116      | 1/4-20 X 1 SOC HD BLK        |
| 1-5  | 900116      | POSITION BELT ATTACHMENT KIT |

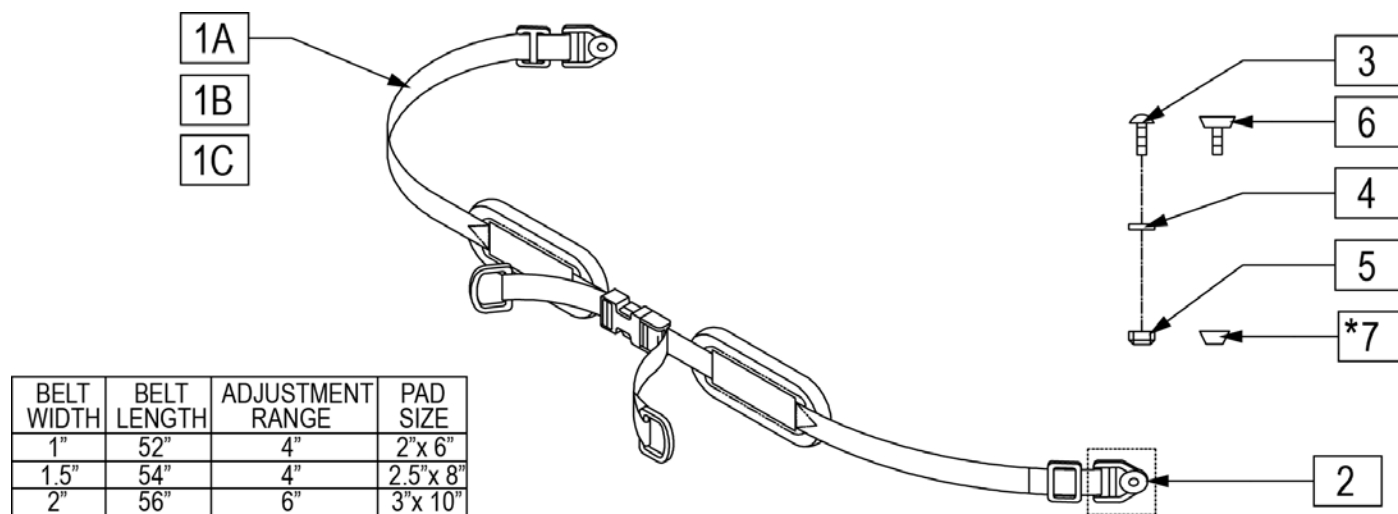


NOTE : ITEM 7 CHANNEL NUT IS USED WHEN ORDERING IN CONJUNCTION WITH CUSTOM CONVENTIONAL SEATING

| Pos.     | Item Number | Description                      |
|----------|-------------|----------------------------------|
| 1A       | 450420-QU   | BELT 1" DUAL ADJ. SQZ BKL PAD    |
| 1B       | 450421-QU   | BELT 1.5" DUAL ADJ. SQZ BKL PAD  |
| 1C       | 450422-QU   | BELT 2" DUAL ADJ. SQZ BKL PAD    |
| 2        | 451116-QU   | PLASTIC WEBBING GROMMET          |
| 3        | 455036-QU   | 1/4-28 x 9/16 SPEC ROUND HD      |
| 4        | 106943      | WASHER FLAT AN 1/4 ROHS          |
| 5        | 479020      | CHAN.UNF NUT 1/4-28 W/SETSCRW    |
| 6        | 106935      | NUT NYLOCK THIN 1/4-28 ROHS      |
| 7        | 481106-QU   | 1/4-28 CHANNELNUT                |
| 1A,2,4-6 | 476520      | BELT 1" DUAL ADJ SQZ BKLPAD AC   |
| 1B,2,4-6 | 476521      | BELT 1.5" DUAL ADJ SQZ BKLPAD AC |
| 1C,2,4-6 | 476522      | BELT 2" DUAL ADJ SQZ BKLPAD AC   |
| 1A-5     | 468420      | BELT 1" DUAL ADJ.SQZ BKLPAD CZ   |
| 1B-5     | 468421      | BELT 1.5" DUAL ADJ.SQZ BKLPAD CZ |
| 1C-5     | 468422      | BELT 2" DUAL ADJ.SQZ BKLPAD CZ   |

**NOTE:** WHEN ADDING A TRAY TO AN EXISTING CHAIR WITH DESK OR FULL LENGTH ARM PADS, ADD 1" TO CURRENT SEAT WIDTH TO ACHIEVE THE BEST TRAY SIZE.

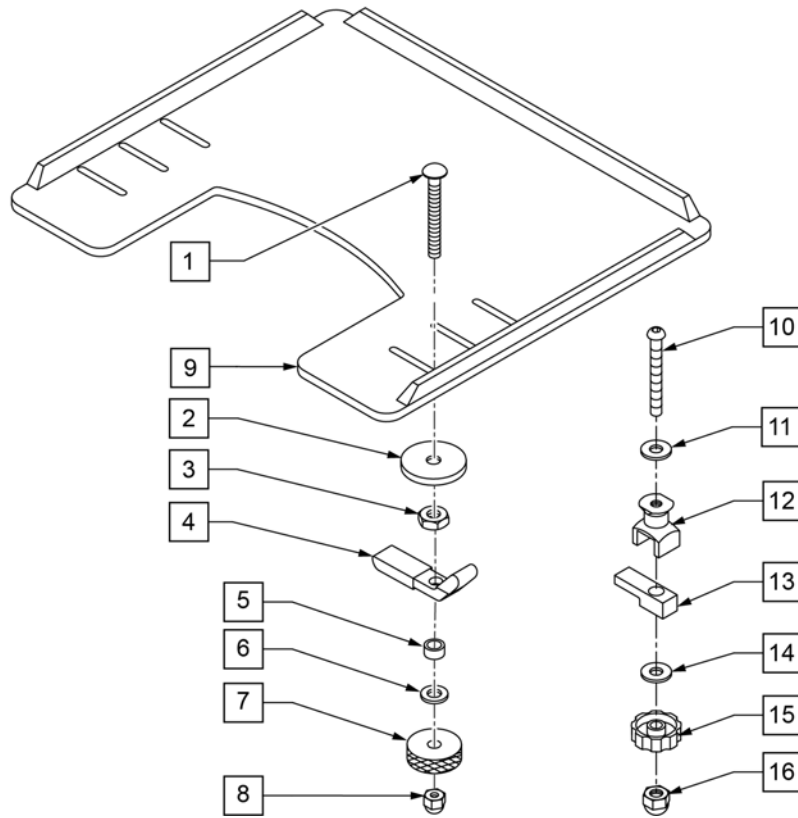

## MEASURING GUIDE



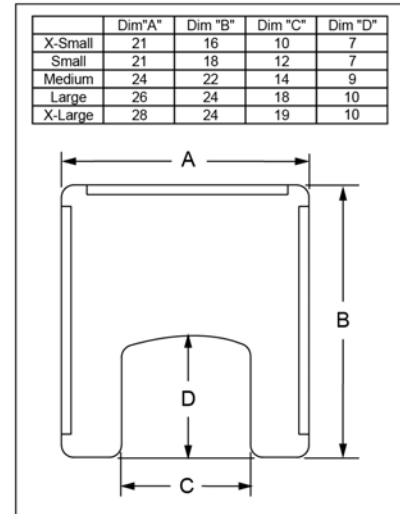


| Pos.     | Item Number | Description                          | Remarks                  |
|----------|-------------|--------------------------------------|--------------------------|
| 1        | 226199      | 5/16-18 x 3 1/2 CARR BOLT            |                          |
| 2        | 263112      | 5/16 WASHER FENDER                   |                          |
| 3        | 202110      | 5/16-18 NUT THIN                     |                          |
| 4        | 550400      | TRAY CLAMP                           |                          |
| 5        | 481113      | SPACER 5/16 X .380                   |                          |
| 6        | 263100      | 5/16 AN WASHER FLAT                  |                          |
| 7        | 483026      | KNURLED TRAY KNOB                    |                          |
| 8        | 202112      | 5/16-18 ACORN NUT                    |                          |
| 1-8      | 881051      | TRAY HARDWARE KIT TUFFAK NEW STYLE   | Pair                     |
| 9A       | 562619      | TRAY TOP EX-SM W/RAIL                |                          |
| 9B       | 562620      | TRAY TOP SM W/RAIL                   |                          |
| 9C       | 562621      | TRAY TOP MED W/RAIL                  |                          |
| 9D       | 562622      | TRAY TOP LRG W/RAIL                  |                          |
| 9E       | 562623      | TRAY TOP X-LRG W/RAIL                |                          |
| 10-15    | 881053      | TRAY HARDWARE ADJ ARM PWR (14219)    | Pair                     |
| 1-8,9A   | 902306      | TRAY ASSEMBLY EX-SM TUFFAK (10 -12") | Fits 10"-12" wide chairs |
| 1-8,9B   | 902303      | TRAY ASSEMBLY SM TUFFAK (13 -14")    | Fits 13"-14" wide chairs |
| 1-8,9C   | 902304      | TRAY ASSEMBLY MED TUFFAK (15 -17")   | Fits 15"-17" wide chairs |
| 1-8,9D   | 902305      | TRAY ASSEMBLY LRG TUFFAK (18 -20")   | Fits 18"-20" wide chairs |
| 1-8,9E   | 902309      | TRAY ASSEMBLY X-LRG TUFFAK (21 -22") | Fits 21"-22" wide chairs |
| 9A,10-15 | 902330      | TRAY ASSM X-SM TFK 10-12/CAM         | Fits 10"-12" wide chairs |

| Pos.     | Item Number | Description                    | Remarks                  |
|----------|-------------|--------------------------------|--------------------------|
| 9B,10-15 | 902331      | TRAY ASSM SM TUFFAK 13-14/CAM  | Fits 13"-14" wide chairs |
| 9C,10-15 | 902332      | TRAY ASSM MED TUFFAK 15-17/CAM | Fits 15"-17" wide chairs |
| 9D,10-15 | 902333      | TRAY ASSM LRG TUFFAK 18-20/CAM | Fits 18"-20" wide chairs |


# TRAY HARDWARE

(04/2012)

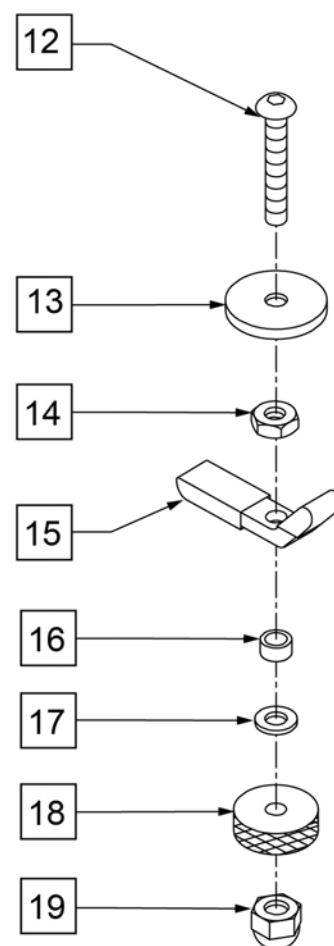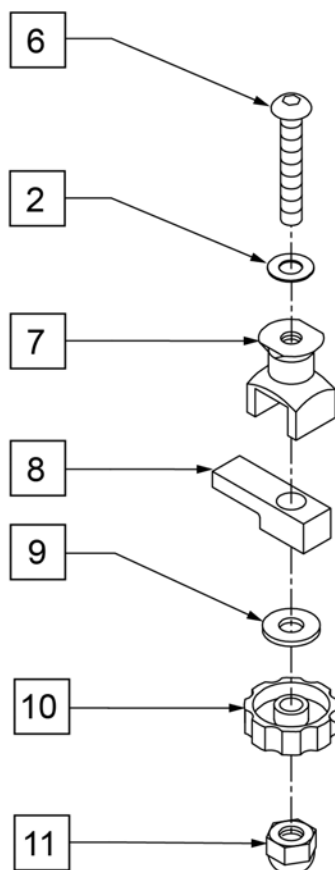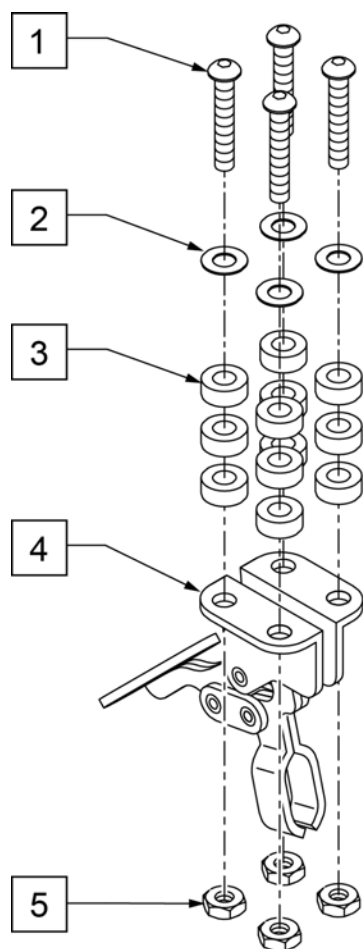


| Pos.  | Item Number | Description                          | Remarks |
|-------|-------------|--------------------------------------|---------|
| 1     | 2249        | 5/16-18 x 1 1/2 BH C/S BLK           |         |
| 2     | 2709        | .625 x .135 x .031 NYLON WASHER      |         |
| 3     | 2710        | .609 x .250 x .250 NYLON SPACER      |         |
| 5     | 202110      | 5/16-18" NUT THIN                    |         |
| 6     | 2021        | 5/16-18" x 2 3/4" BUTTON HD C/S      |         |
| 7     | 14240       | TRAY BUSHING                         |         |
| 8     | 14230       | TRAY LOCK BAR                        |         |
| 9     | 2705        | 1/4" FLAT WASHER                     |         |
| 10    | 14220       | TRAY KNOB                            |         |
| 11    | 202114      | 5/16-18" ACORN LOCK NUT              |         |
| 12    | 226199      | 5/16-18" x 3 1/2" CARR BOLT          |         |
| 13    | 263112      | 5/16 WASHER FENDER                   |         |
| 14    | 202110      | 5/16-18" NUT THIN                    |         |
| 15    | 550400      | TRAY CLAMP                           |         |
| 16    | 481013      | BEARING SPACER                       |         |
| 17    | 263100      | 5/16" AN WASHER FLAT                 |         |
| 18    | 483026      | KNURLED TRAY KNOB                    |         |
| 19    | 202112      | 5/16-18" ACORN NUT                   |         |
| 1-5   | 4099        | TOGGLE CLAMP - HIDEAWAY              |         |
| 6-11  | 881053      | TRAY HARDWARE ADJ ARM PWR (14219)    |         |
| 12-19 | 881051      | TRAY HDWR KIT TUFFAK NEW STYLE (9QH) | Pair    |

**NOTE:** ONE 19" RIB AND TWO 15 1/2" RIBS FIT THE EXTRA-SMALL, SMALL, 9-0 AND 9-1 TRAY TABLES. ONE 21" RIB AND TWO 19" RIBS FIT THE MEDIUM AND 9-2 TRAY TABLES. ONE 23 1/2" RIB AND TWO 21" RIBS FIT THE LARGE, 9-2, 9-5, AND 9-6 TRAY TABLES.

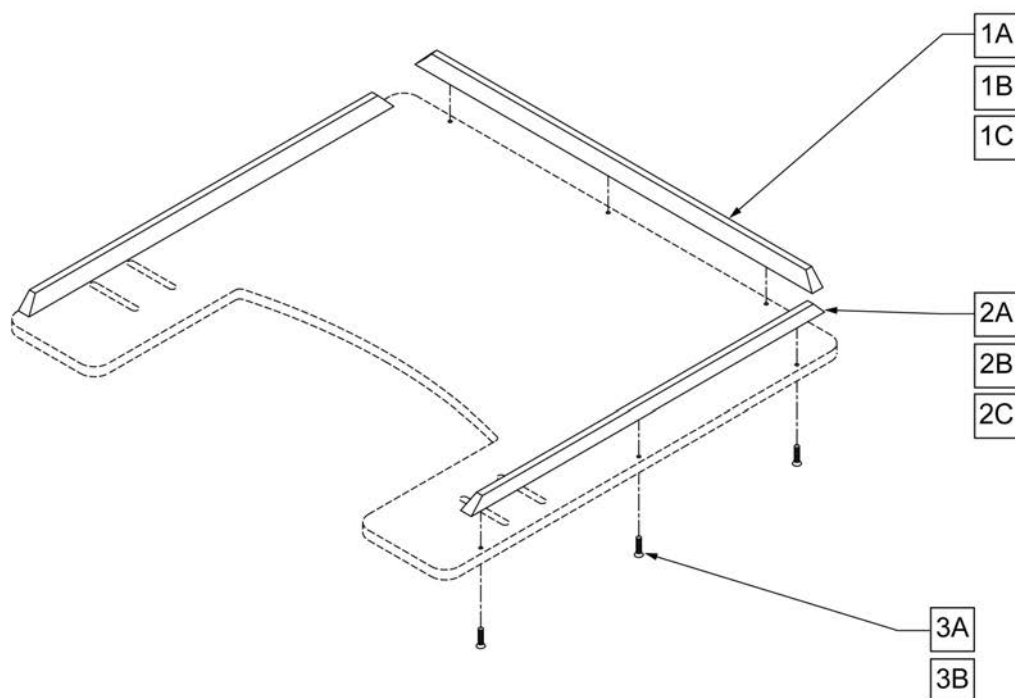


| Pos. | Item Number | Description                            | Pos. | Item Number | Description                            |
|------|-------------|----------------------------------------|------|-------------|----------------------------------------|
| 1A   | 9004        | 19" WOOD TRAY RIBS                     | 2A   | 9004        | 19" WOOD TRAY RIBS                     |
| 1A   | 9005        | 21" WOOD TRAY RIBS                     | 2A   | 9005        | 21" WOOD TRAY RIBS                     |
| 1A   | 9006-QU     | 23 1/2" WOOD TRAY RIBS                 | 2B   | 9007-QU     | 15 1/2" HEAVY DUTY RIB FOR WOOD TRAY   |
| 1B   | 9010        | 19" HEAVY DUTY RIB FOR WOOD TRAY       | 2B   | 9010        | 19" HEAVY DUTY RIB FOR WOOD TRAY       |
| 1B   | 9013        | 21" HEAVY DUTY RIB FOR WOOD TRAY       | 2B   | 9013        | 21" HEAVY DUTY RIB FOR WOOD TRAY       |
| 1B   | 9016        | 23 1/2" HEAVY DUTY RIB FOR WOOD TRAY   | 2C   | 9008-QU     | 15 1/2" HEAVY DUTY RIB FOR 3/8" TUFFAK |
| 1C   | 9011-QU     | 19" HEAVY DUTY RIB FOR 3/8" TUFFAK     | 2C   | 9011-QU     | 19" HEAVY DUTY RIB FOR 3/8" TUFFAK     |
| 1C   | 9014        | 21" HEAVY DUTY RIB FOR 3/8" TUFFAK     | 2C   | 9014        | 21" HEAVY DUTY RIB FOR 3/8" TUFFAK     |
| 1C   | 9017        | 23 1/2" HEAVY DUTY RIB FOR 3/8" TUFFAK | 3A   | 2268        | #6 x 5/8" OVAL PHILLIPS SCREW          |
| 2A   | 9003        | 15 1/2" WOOD TRAY RIBS                 | 3B   | 230047      | #6 x 3/4" FH PHIL S/M SCREW            |




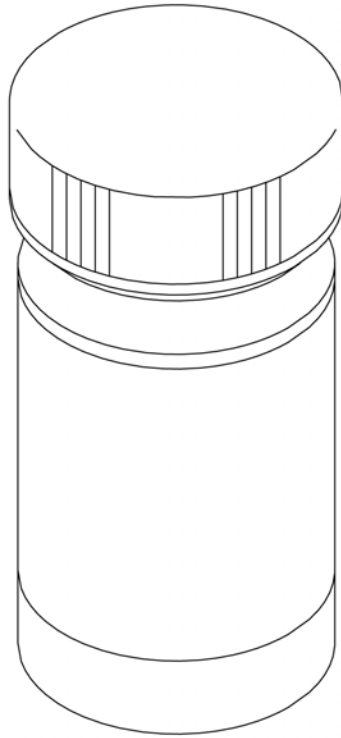

| Pos. | Item Number | Description              |     |
|------|-------------|--------------------------|-----|
| 1    | 160504      | TOUCH UP BLACK           | 1oz |
| 1    | 160510      | TOUCH UP BLUE            | 1oz |
| 1    | 160514      | TOUCH UP YELLOW          | 1oz |
| 1    | 160542      | TOUCH UP MIDNIGHT PURPLE | 1oz |
| 1    | 160551      | TOUCH UP FOREST GREEN    | 1oz |
| 1    | 160555      | TOUCH UP PEARL PINK      | 1oz |
| 1    | 160560      | TOUCH UP TOXIC GREEN     | 1oz |
| 1    | 160561      | TOUCH UP BLUE VELVET     | 1oz |
| 1    | 160562      | TOUCH UP BLACK OPAL      | 1oz |
| 1    | 160563      | TOUCH UP CANDY RED       | 1oz |
| 1    | 160564      | TOUCH UP CANDY BLUE      | 1oz |
| 1    | 160565      | TOUCH UP ORANGE          | 1oz |
| 1    | 160566      | TOUCH UP BLUE OPAL       | 1oz |
| 1    | 160567      | TOUCH UP CANDY TURQOISE  | 1oz |
| 1    | 160568      | TOUCH UP RAZZLEBERRY     | 1oz |

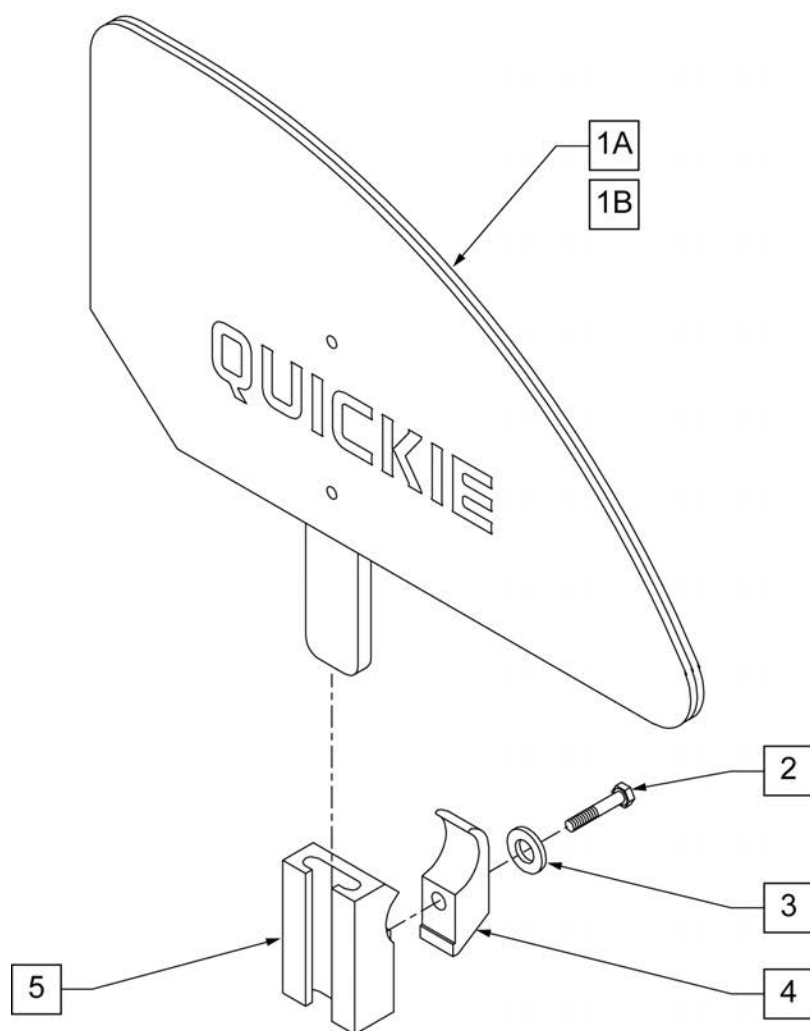


## MEASURING GUIDE

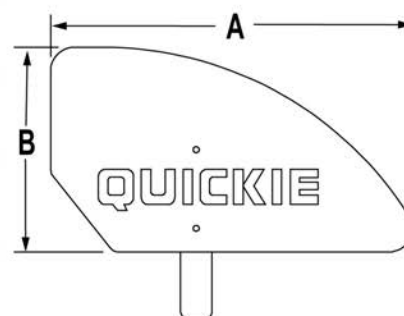

|       | "A"     | "B"    |
|-------|---------|--------|
| ADULT | 12 3/4" | 8 5/8" |
| KIDS  | 11 1/2" | 6 1/2" |


**NOTE:** STANDARD (ADULT) SIDE GUARD FITS 14-20" DEPTHS, KIDS SIDE GUARD FITS 10-13" DEPTHS.

| Pos.   | Item Number | Description                          |
|--------|-------------|--------------------------------------|
| 1A     | 881150      | PLASTIC SIDE GUARD KIDS R            |
| 1A     | 881151      | PLASTIC SIDE GUARD KIDS L            |
| 1B     | 881152      | PLASTIC SIDE GUARD STD R             |
| 1B     | 881153      | PLASTIC SIDE GUARD STD L             |
| 2      | 106971      | 1/4-28 x 1 3/8 HEX C/S               |
| 3      | 106944      | 1/4 AN WASHER FLAT THIN              |
| 4      | 550251      | CLAMP P300 CHARGING/ADJ FL KIDS      |
| 5      | 550325      | SIDE GUARD/KIDZ ARM RECEIVER         |
| 2-5    | 881200      | SIDE GUARD/KIDZ ARM RECEIVER W/CLAMP |
| 1A,2-5 | 901111      | PLASTIC SIDE GUARD KIDS ASSM W/CLAMP |
| 1B,2-5 | 901110      | PLASTIC SIDE GUARD STD ASSM W/CLAMP  |


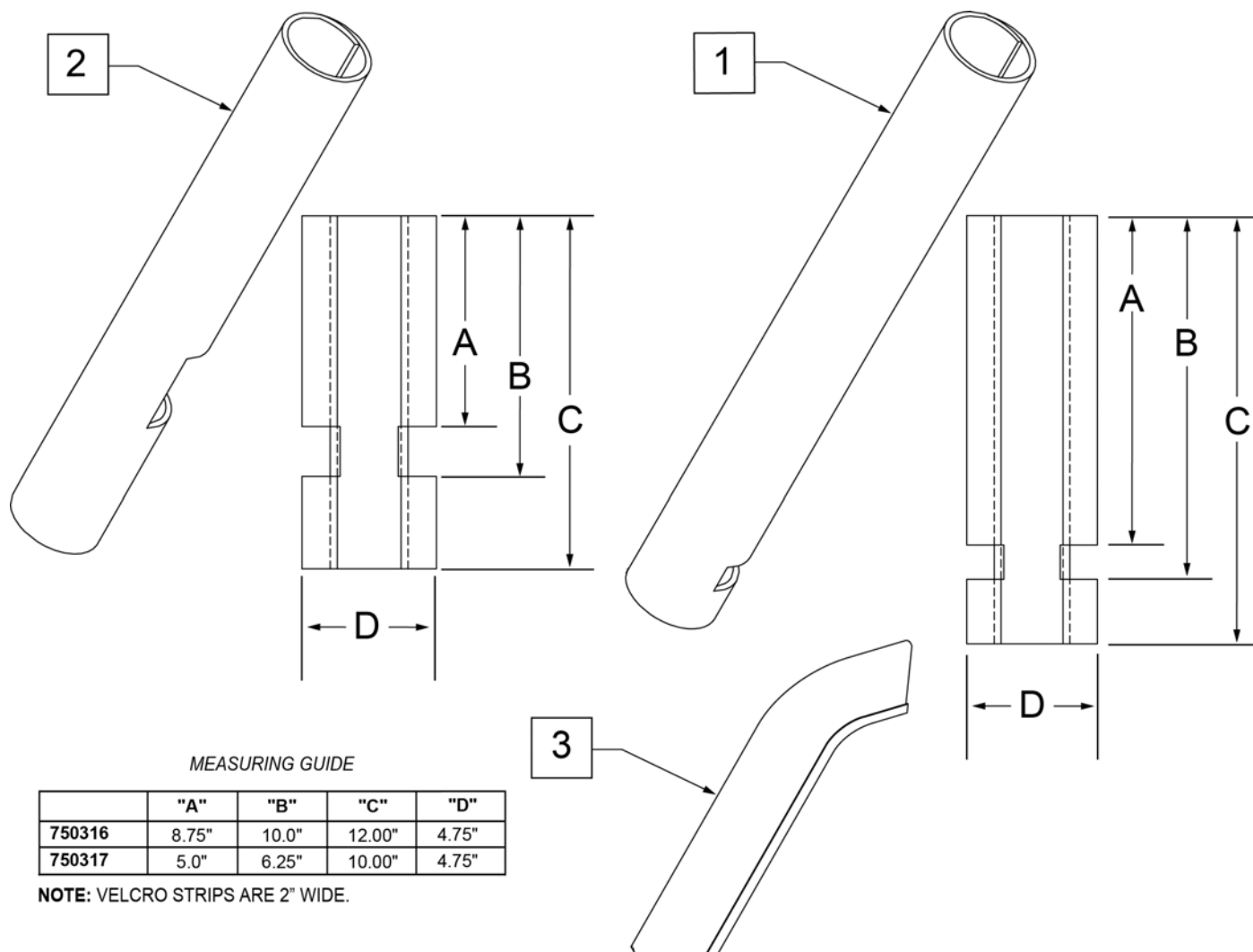


| Pos. | Item Number | Description                       | Remarks                            |
|------|-------------|-----------------------------------|------------------------------------|
| 1    | 750316      | IMPACT GUARDS NEOPRENE FIXED (PR) | PAIR-Fixed Front Frame             |
| 2    | 750317      | IMPACT GUARDS NEOPRENE S/A        | PAIR-S/A Footrest & Q7             |
| 3    | 109231      | GEL VELCRO PAD HANGER/FRAME       | PAIR- fits S/A & Fixed Front Frame |


**NOTE:** KIDS BACKPACKS DO NOT HAVE ADJUSTABLE STRAPS W/ BUCKLES.

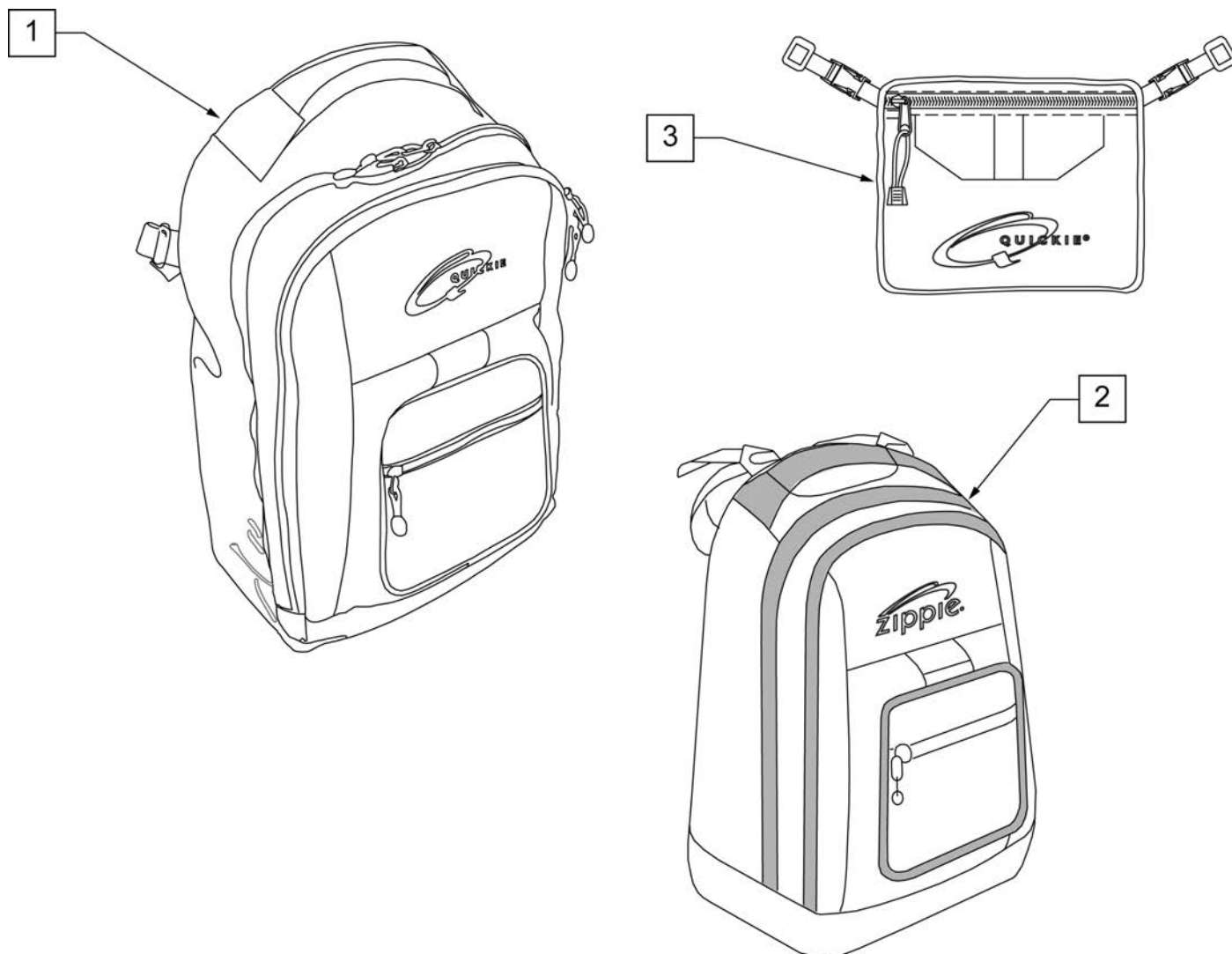


| Pos. | Item Number | Description        |
|------|-------------|--------------------|
| 1    | 750115      | BACK PACK QUICKIE  |
| 2    | 104572      | BACKPACK ZIPPIE    |
| 3    | 750307      | SEAT POUCH QUICKIE |

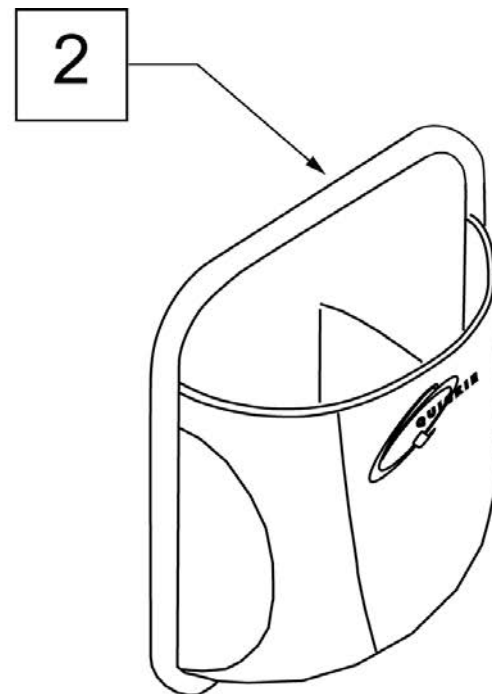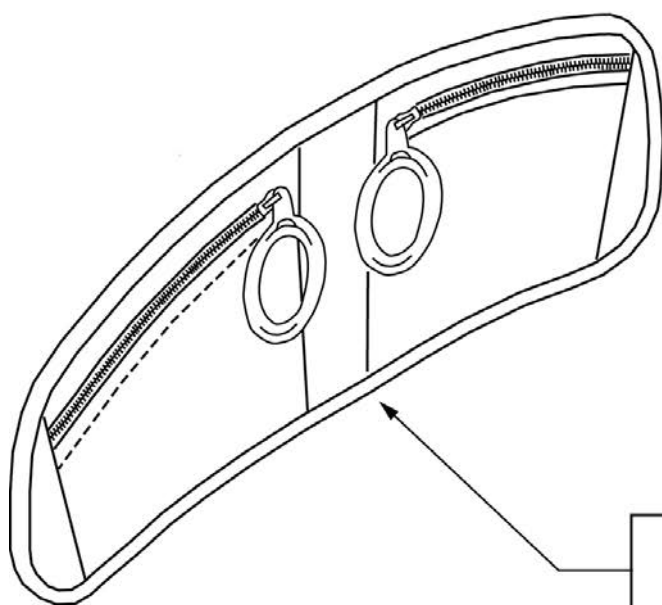


| Pos. | Item Number | Description         | Remarks                   |
|------|-------------|---------------------|---------------------------|
| 1    | 013477      | LAP ORGANIZER       | Size: 18.25"x 7.75"x .75" |
| 2    | 013475      | QUICKIE SIDE POCKET | Size: 6.5"x 6.5"x 2"      |

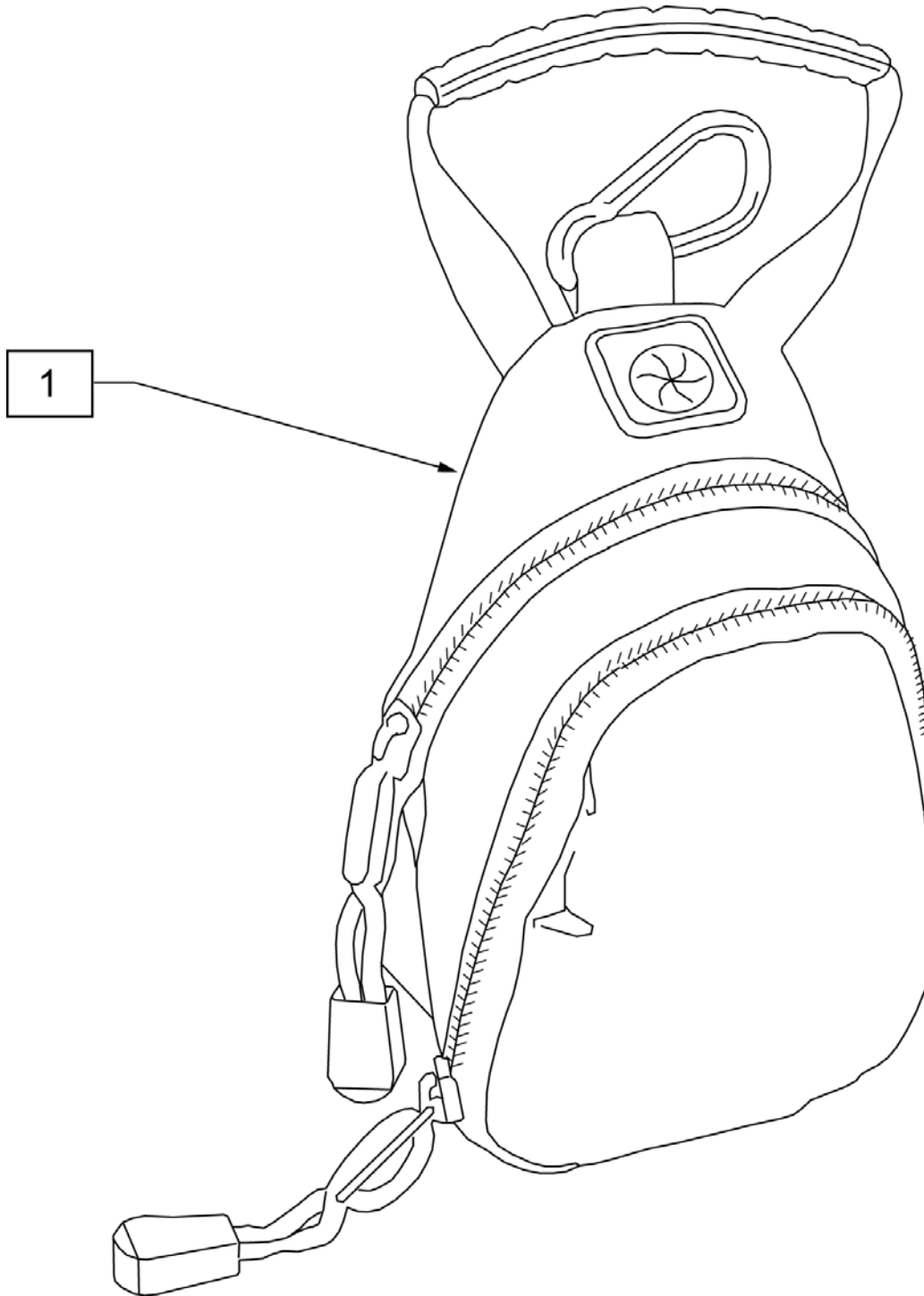

| Pos. | Item Number | Description              |
|------|-------------|--------------------------|
| 1    | 109229      | CELL PHONE PACK (ORANGE) |

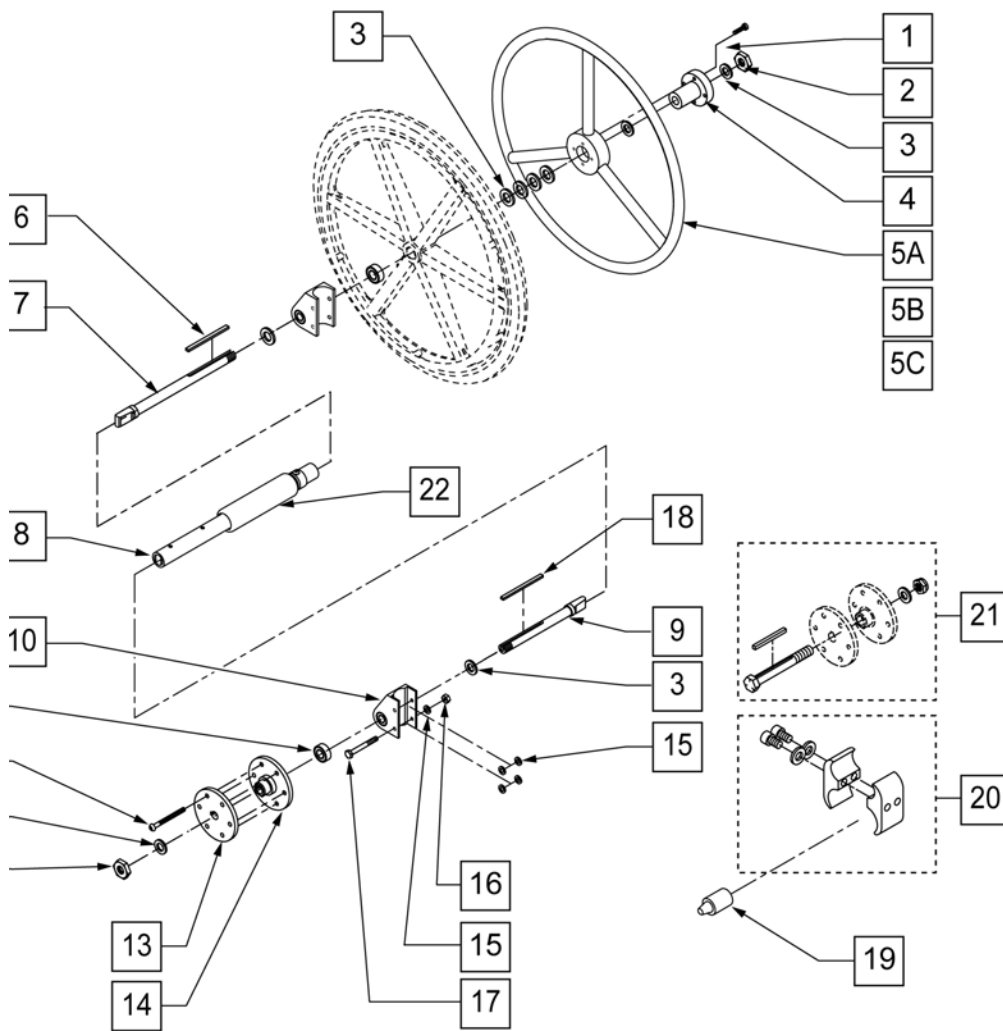


## MEASURING GUIDE

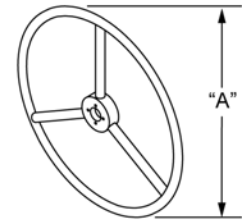


| WHEEL SIZE | STD "A" | SMALL "A" |
|------------|---------|-----------|
| 20"        | 16 1/2" | -         |
| 22"        | 18"     | 16 1/2"   |
| 24"        | 19 1/4" | 18"       |

## UNLOCKED - OPEN

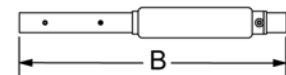

## LOCKED - CLOSED

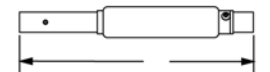


| CHAIR WIDTH | "B"     | "C" |
|-------------|---------|-----|
| 13"         | 9 3/4"  | 8"  |
| 14"         | 10 3/4" | 9"  |
| 15"         | 11 3/4" | 10" |
| 16"         | 12 3/4" | 11" |
| 17"         | 13 3/4" | 12" |
| 18"         | 14 3/4" | 13" |
| 19"         | 15 3/4" | 14" |
| 20"         | 16 3/4" | 15" |


| Pos. | Item Number | Description                   | Remarks |
|------|-------------|-------------------------------|---------|
| 1    | 106956      | 10-32 x 3/4 SOC C/S BLK       |         |
| 2    | 106940      | 1/2-20 NYLOCK NUT             |         |
| 3    | 106949      | 1/2 AN WASHER FLAT            |         |
| 4    | 106691      | 1 AD HUB CENTER               |         |
| 5A   | 529801      | 1 AD HR ONLY 20" STD/22"SM    |         |
| 5B   | 529800      | 1 AD HR ONLY 22" STD/24"SM    |         |
| 5C   | 529802      | 1 AD HR ONLY 24" STD          |         |
| 6    | 341274      | 3/16 SQ x 1-1/2 KEY STL       |         |
| 7    | 481131      | 1 AD HANDRIM DRIVE SHAFT      |         |
| 8    | 101097-001  | 1 AD SHAFT ASSM 12"           |         |
| 8    | 101097-002  | 1 AD SHAFT ASSM 13"           |         |
| 8    | 101097-003  | 1 AD SHAFT ASSM 14"           |         |
| 8    | 101097-004  | 1 AD SHAFT ASSM 15"           |         |
| 8    | 101097-005  | 1 AD SHAFT ASSM 16"           |         |
| 8    | 101097-006  | 1 AD SHAFT ASSM 17"           |         |
| 8    | 101097-007  | 1 AD SHAFT ASSM 18"           |         |
| 9    | 481132      | 1 AD WHEEL DRIVE SHAFT        |         |
| 10   | 880834      | 1 AD AXLE PLATE W/BEARINGS LX |         |

| Pos.        | Item Number | Description                              | Remarks |
|-------------|-------------|------------------------------------------|---------|
| 11          | 483058      | MAG AXLE SPACER W/HOLE 1 AD WHEEL SPACER | LEFT    |
| 12          | 107227      | 1/4-28 x 2 1/2 BH C/S BLK                |         |
| 13          | 500630      | 1 AD WHEEL HUB W/O TAP                   |         |
| 14          | 500631      | WHEEL HUB 1 ARM DRIVE W/TAP              |         |
| 15          | 106944      | 1/4 AN WASHER FLAT THIN                  |         |
| 16          | 106934      | 1/4-28 NYLOCK NUT                        |         |
| 17          | 106973      | 1/4-28 x 2 G8 HEX C/S                    |         |
| 18          | 341277      | 3/16" SQ x 2-3/8" KEY                    |         |
| 19          | 482030      | 1 AD DRIVE RECEIVER                      |         |
| 20          | 107639      | WHL LCK CLMP ASM                         |         |
| 21          | 330015      | 1AD ALIGNMENT TOOL ASSM                  |         |
| 22          | 101109      | 1 AD COVER ASSM                          |         |
| 23          | 483057      | MAG AXLE SPACER W/HOLE 1 AD WHEEL SPACER | RIGHT   |
| 2,3,7,9-19  | 881081      | 1 AD HDWR KIT TS                         |         |
| 1-4,5A,6-20 | 116616      | 1AD 12 RTRO 20 STD/22SM HR TS            |         |
| 1-4,5A,6-20 | 923780      | 1AD 13" RETRO 20 STD/22SM HR TS          |         |
| 1-4,5A,6-20 | 923781      | 1AD 14" RETRO 20 STD/22SM HR TS          |         |
| 1-4,5A,6-20 | 923782      | 1AD 15" RETRO 20 STD/22SM HR TS          |         |
| 1-4,5A,6-20 | 923783      | 1AD 16" RETRO 20 STD/22SM HR TS          |         |
| 1-4,5B,6-20 | 116617      | 1AD 12 RTRO 22 STD/24SM HR TS            |         |
| 1-4,5B,6-20 | 923788      | 1AD 13" RETRO 22 STD/24SM HR TS          |         |
| 1-4,5B,6-20 | 923789      | 1AD 14" RETRO 22 STD/24SM HR TS          |         |
| 1-4,5B,6-20 | 923790      | 1AD 15" RETRO 22 STD/24SM HR TS          |         |
| 1-4,5B,6-20 | 923791      | 1AD 16" RETRO 22 STD/24SM HR TS          |         |
| 1-4,5C,6-20 | 116618      | 1AD 12 RTRO 24 STD HR TS                 |         |
| 1-4,5C,6-20 | 923794      | 1 AD 13" RETRO 24 STD HR TS              |         |
| 1-4,5C,6-20 | 923795      | 1 AD 14" RETRO 24 STD HR TS              |         |
| 1-4,5C,6-20 | 923796      | 1 AD 15" RETRO 24 STD HR TS              |         |
| 1-4,5C,6-20 | 923797      | 1 AD 16" RETRO 24 STD HR TS              |         |
| 1-4,5A,6-20 | 923850      | 1 AD 17" RETRO 20 STD/22 SM HR TS        | RIGHT   |
| 1-4,5A,6-20 | 923853      | 1 AD 18" RETRO 20 STD/22 SM HR TS        | LEFT    |
| 1-4,5B,6-20 | 923851      | 1 AD 17" RETRO 22 STD/24 SM HR TS        | RIGHT   |
| 1-4,5B,6-20 | 923854      | 1 AD 18" RETRO 22 STD/24 SM HR TS        | LEFT    |
| 1-4,5C,6-20 | 923852      | 1 AD RETRO 24 STD HR TS                  | RIGHT   |
| 1-4,5C,6-20 | 923855      | 1 AD RETRO 24 STD HR TS                  | LEFT    |




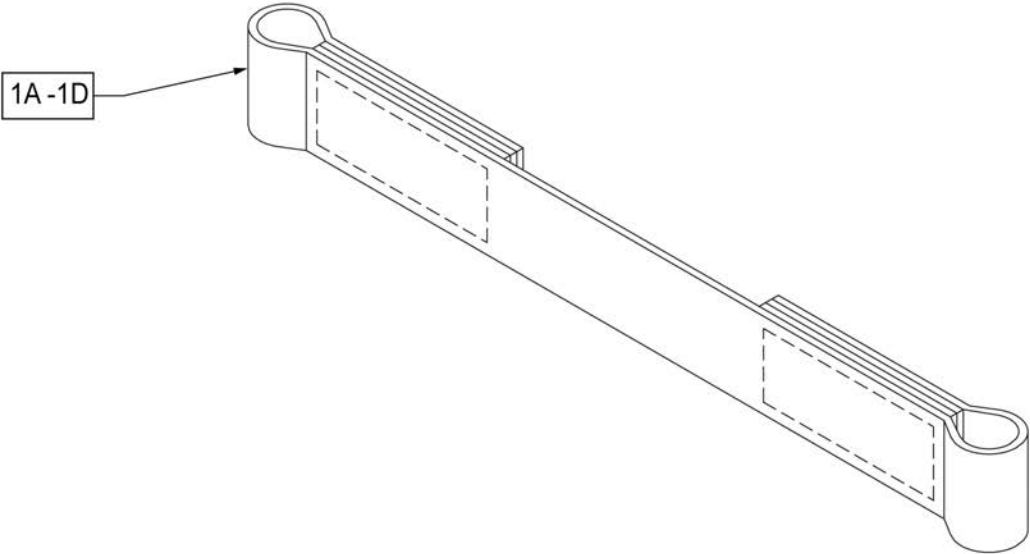

| Pos. | Item Number | Description                   |
|------|-------------|-------------------------------|
| 1A   | 109356      | LEG STRAP SOFT FEEL Q 11 -12" |
| 1B   | 109357      | LEG STRAP SOFT FEEL Q 13 -15" |
| 1C   | 109366      | LEG STRAP SOFT FEEL Q 16 -18" |
| 1D   | 111487      | LEG STRAP SOFT FEEL Q 19 -20" |


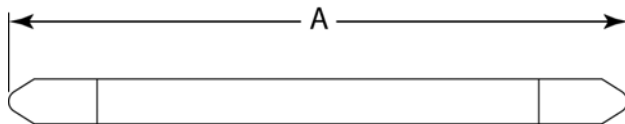
**MEASURING GUIDE**

| DESCRIPTION           | DIM "A" |
|-----------------------|---------|
| LEG STRAP 11"-12" BLK | 27 1/2  |
| LEG STRAP 13"-15" BLK | 32 1/2  |
| LEG STRAP 16"-18" BLK | 39 1/2  |
| LEG STRAP 19"-20" BLK | 45 1/2  |

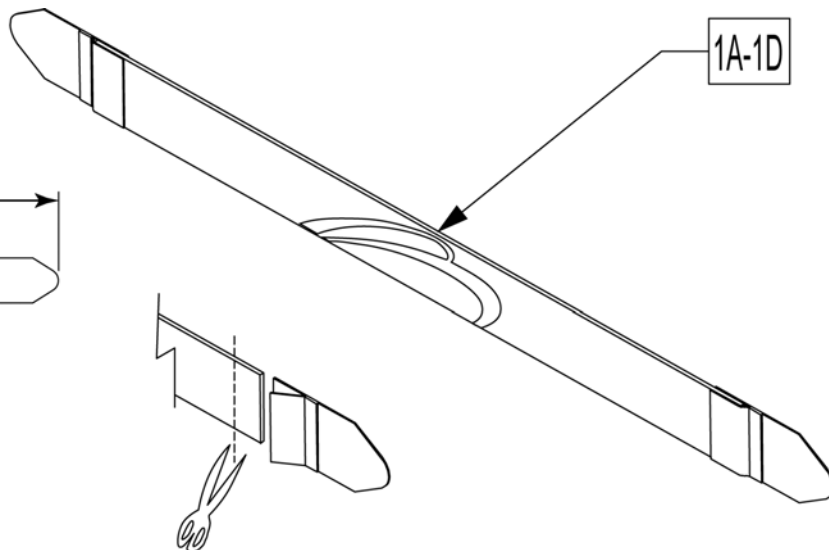

Leg Straps may be custom cut to a smaller size


| Pos. | Item Number | Description           |
|------|-------------|-----------------------|
| 1A   | 113255      | LEG STRAP 11"-12" BLK |
| 1B   | 113256      | LEG STRAP 13"-15" BLK |
| 1C   | 113257      | LEG STRAP 16"-18" BLK |
| 1D   | 113258      | LEG STRAP 19"-20" BLK |

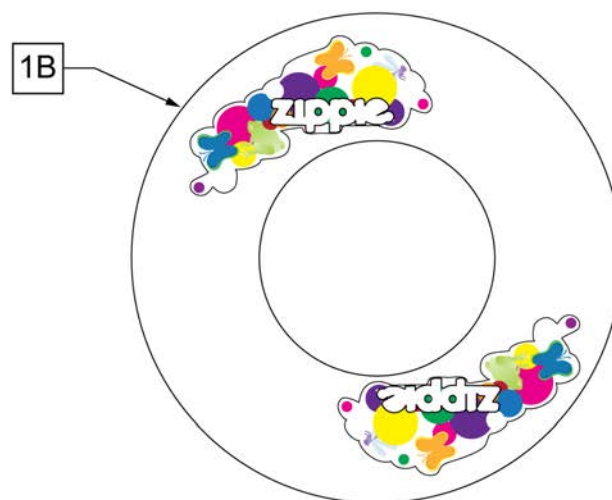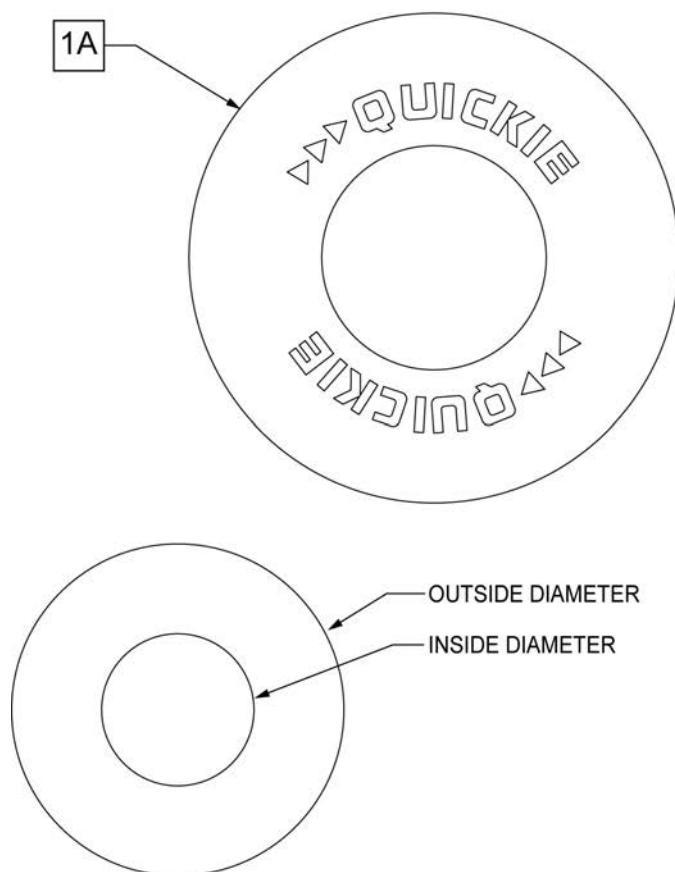


SPOKE GUARD MEASURING GUIDE

| PART NUMBER | INSIDE DIAMETER | OUTSIDE DIAMETER |
|-------------|-----------------|------------------|
| 902001      | 8 1/4           | 15 7/8           |
| 902011      | 8 1/4           | 18 1/8           |
| 902021      | 10 1/4          | 19 5/8           |
| 902025      | 10 1/4          | 20 3/8           |
| 103660-001  | 8 1/4           | 16               |
| 103660-002  | 8 1/4           | 18               |
| 103663-001  | 10 1/4          | 20 1/2           |
| 103663-002  | 10 1/4          | 23 1/2           |

| Pos. | Item Number | Description                    | Remarks                                      |
|------|-------------|--------------------------------|----------------------------------------------|
| >>>> |             | BLACK SPOKE GUARDS             |                                              |
| 1A   | 902001      | SPK GRDS 20" QUICKIE BLK       | Pair-BLACK-FITS 20" WHEELS                   |
| 1A   | 902011      | SPK GRDS 22" QUICKIE BLK       | Pair-BLACK-FITS 22" WHEELS                   |
| 1A   | 902021      | SPK GRD 24" QUICKIE SCR MT BLK | Pair-BLACK-FITS 24" (540) MAG & SPOKE WHEELS |
| 1A   | 902025      | SPK GRD 24" QUICKIE TAB MT BLK | Pair-BLACK-FITS 25" (559) SPOKE WHEELS       |
| >>>> |             | CLEAR SPOKE GUARDS             |                                              |
| 1B   | 112644-001  | SPOKE GUARD ZIPPIE 20"         | Pair-CLEAR-FITS 20" WHEELS                   |
| 1B   | 112644-002  | SPOKE GUARD ZIPPIE 22"         | Pair-CLEAR-FITS 22" WHEELS                   |
| 1B   | 112644-003  | SPOKE GUARDZIPPIE 24" MAG      | Pair-CLEAR-FITS 24" (540) MAG WHEELS         |
| 1B   | 112644-004  | SPOKE GUARD ZIPPIE 24" SPOKE   | Pair-CLEAR-FITS 24" (540) SPOKE WHEELS       |

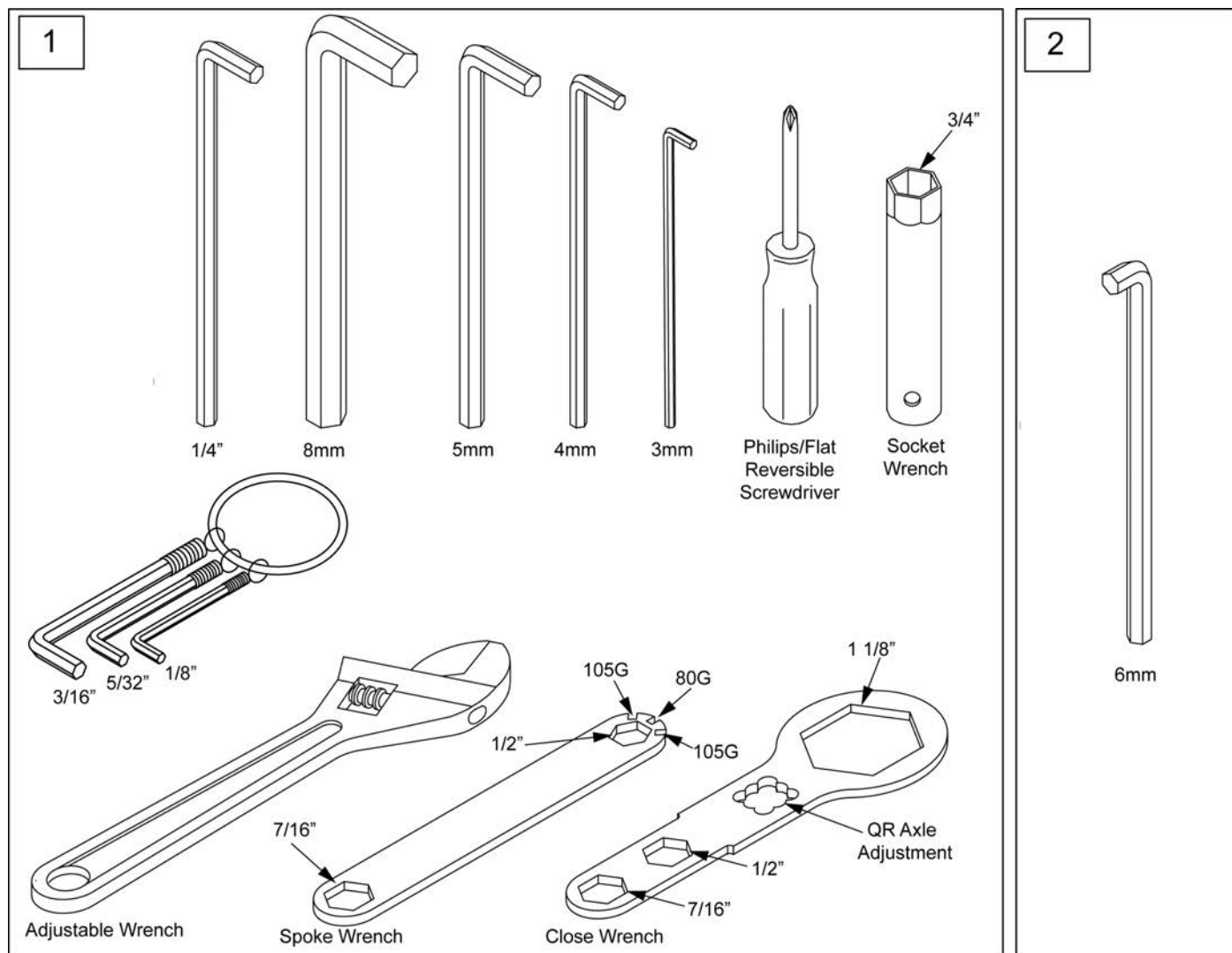

| Pos. | Item Number | Description                   | Remarks                   |
|------|-------------|-------------------------------|---------------------------|
| 1    | 108314      | QUICKIE COMBO TOOL KIT        | Includes pouch            |
| 2    | 113403      | 6MM ALLEN WRENCH L, 7MM SHORT | caster spline chairs only |

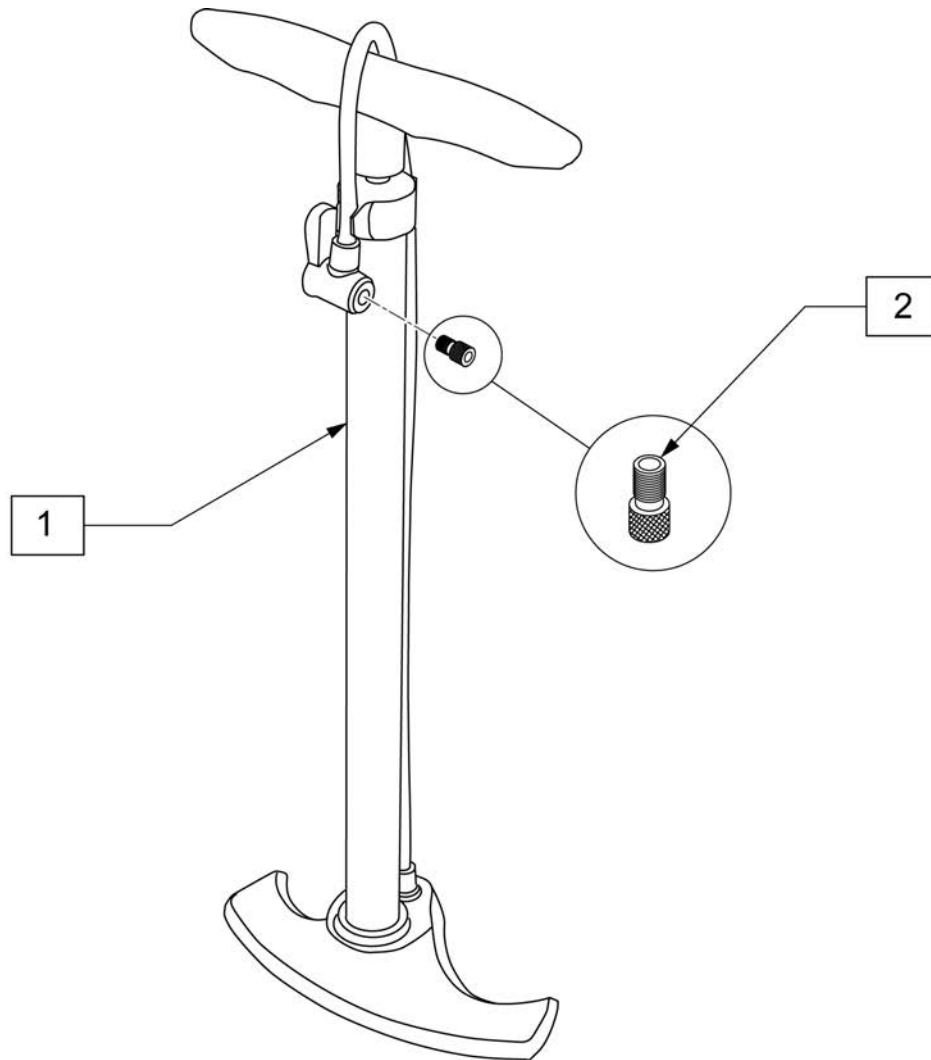

| Pos. | Item Number | Description          |
|------|-------------|----------------------|
| 1    | 389050      | TIRE PUMP            |
| 2    | 381230      | VALVE ADAPTER PRESTA |



# **NEW ZIPPIE® TS**