

# Merlin

## Bathseat



READ THESE INSTRUCTIONS CAREFULLY BEFORE FITTING A BATHSEAT.

The purpose of a bathseat is to provide an intermediary support in the conventional bath as an aid to bathing for those who have difficulties in lowering or raising themselves fully into or out of a bath.


These instructions apply to the following bathseats:

Product Nos: 10157C • 10158C • 10159C  
Maximum user weight: 160kg (25st)

### FITTING INSTRUCTIONS

1. Place the bathseat into the bath.
2. Push down on the bathseat to ensure the suction pads are secured to the bottom of the bath (Fig. 1).
3. Fill the bath to the desired level.
4. Ensure that the bathseat is stable and the suction pads are secured before use.
5. To remove, lift the suction pad tabs to release and lift out slowly allowing the water to drain.

Fig.1

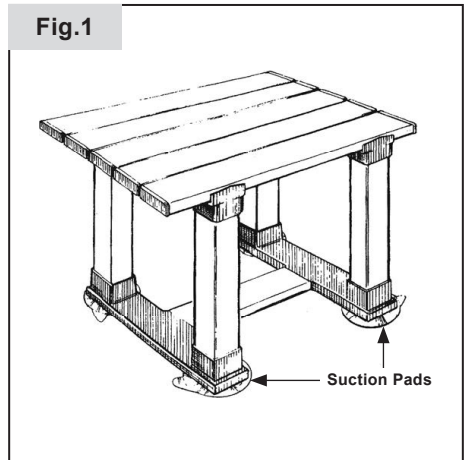

### CLEANING & CARE

All parts may be cleaned using a mild detergent or warm soapy water. Rinse in warm water after cleaning.





### WARNING







Bath oils and other bath time moisturisers may impair the adhesion of the suction pads.

Always check the bathseat is stable and the suckers are secure before use.

# LABELLING / NAMEPLATE

# EXAMPLE

|   |   |  |
|---|---|--|
|               | Sunrise Medical Ltd. Limited<br>Thorns Road, Brierley Hill<br>West Midlands DY5 2LD<br>United Kingdom | <div style="border: 1px solid black; padding: 2px; display: inline-block;">                     EU REP                 </div>  |
| <b>BATHSEAT</b>   |  <b>2025-01-16</b>   | James Leckey Design Limited<br>19 Balthamsey Road<br>Lisburn BT28 2SA,<br>Northern Ireland   |
| TYPE: Merlin Bath Seat 152mm (6")   |   |  |
| PLASTIC SLATTED BATH SEAT: 10157C   |   | <div style="border: 1px solid black; padding: 2px; display: inline-block;">                     LOT                 </div> <b>210250300050</b>   |
|  MAX<br>160kg |                      | <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="border: 1px solid black; padding: 2px; display: inline-block;">                     MD                 </div> <div style="border: 1px solid black; padding: 2px; display: inline-block;">                     UK<br/>CA                 </div> <div style="font-size: 24px; font-weight: bold;">                     CE                 </div> </div> |

- |  |   |
|--|---|
| <b>TYPE:</b>   | Product Name/SKU Number                         |
|             | Load Maximum                                    |
|             | Product height                                  |
| <div style="border: 1px solid black; padding: 2px; display: inline-block;">UK<br/>CA</div>   | UKCA Mark                                       |
| <div style="font-size: 24px; font-weight: bold;">CE</div>                                    | CE Mark   |
|             | Consult instructions for use.                   |
|             | LATEX FREE - indicates product is free of Latex |
|  XXXX-XX-XX | Date of manufacture                             |
| <div style="border: 1px solid black; padding: 2px; display: inline-block;">SN</div>          | Serial number                                   |
| <div style="border: 1px solid black; padding: 2px; display: inline-block;">LOT</div>         | LOT number                                      |
| <div style="border: 1px solid black; padding: 2px; display: inline-block;">MD</div>          | This symbol means Medical Device                |
|             | Manufacturer's address                          |
| <div style="border: 1px solid black; padding: 2px; display: inline-block;">EU REP</div>      | European Authorized Representative              |



**ISO 7010-M002**  
**Instruction manual/booklet must be read! (Blue Icon)**



**UK (including retailers, NHS, and mobility stores):**  
 Sunrise Medical Customer Services  
 Sunrise Medical  
 Thorns Road  
 Brierley Hill  
 West Midlands DY5 2LD  
 England  
 Phone: 0845 605 66 88  
 Fax: 0845 605 66 89  
 www.SunriseMedical.co.uk



**Online Purchases or EU Customers:**  
 Helping Hand Customer Services  
 The Helping Hand Company  
 Bromyard Road  
 Ledbury  
 Herefordshire  
 HR8 1NS  
 United Kingdom  
 Tel: +44 (0)1531 635678  
 E-mail: sales@helpinghand.co.uk  
 helping-hand.com

CE

Conforms to  
 The Medical Device Regulation  
 (EU) MDR 2017/745

MD

As the manufacturer, SUNRISE MEDICAL, declares that the product conforms to the UK Medical Devices Regulation 2002 No. 618.