




READ THESE INSTRUCTIONS CAREFULLY
BEFORE FITTING A BATHSEAT.

The purpose of a bathseat is to provide an intermediary support in the conventional bath as an aid to bathing for those who have difficulties in lowering or raising themselves fully into or out of a bath.

These instructions apply to the following bathseats:

Product Nos: 10157 • 10158 • 10159

Maximum user weight: 160kg (25st)

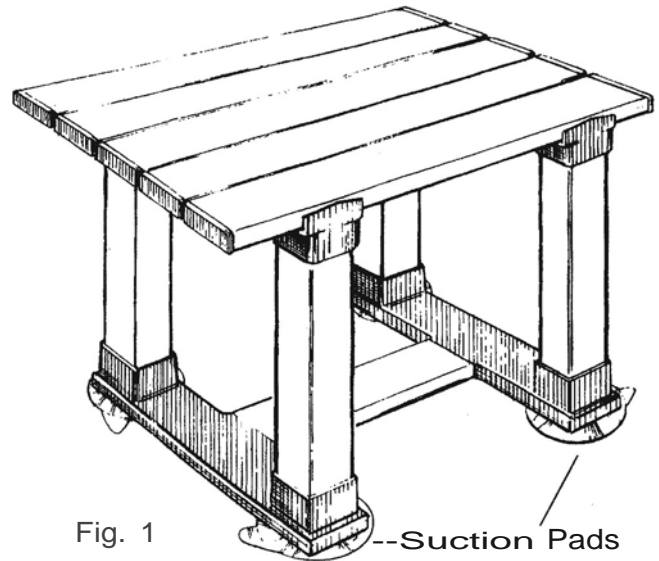

FITTING INSTRUCTIONS

- 1 Place the bathseat into the bath.
- 2 Push down on the bathseat to ensure the suction pads are secured to the bottom of the bath (Fig. 1).
- 3 Fill the bath to the desired level.
- 4 Ensure that the bathseat is stable and the suction pads are secured before use.
- 5 To remove, lift the suction pad tabs to release and lift out slowly allowing the water to drain.

CLEANING & CARE

All parts may be cleaned using a mild detergent or warm soapy water. Rinse in warm water after cleaning.

WARNING

Bath oils and other bath time moisturisers may impair the adhesion of the suction pads.

Always check the bathseat is stable and the suckers are secure before use.



CE

Sunrise Medical Limited
Thorns Road, Brierley Hill.
West Midlands. DY5 2LD
Tel: +44 (0) 845 605 66 88
Fax: +44 (0) 845 605 66 89
orders@sunmed.co.uk
www.sunrisemedical.com