Jay® Lite

IMPORTANT CONSUMER INFORMATION

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WARNING!

- Installing a cushion on a wheelchair may affect the centre of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury.
- Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

All Jay cushions are designed to help reduce the incidence of decubitus ulcers. However, no cushion can completely eliminate sitting pressure or prevent pressure sores. Jay cushions are not a substitute for good skin care including; proper diet, cleanliness and regular weight transfers. In addition, prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue break down is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.

Avoid sharp objects or exposure to excessive heat or open flame.

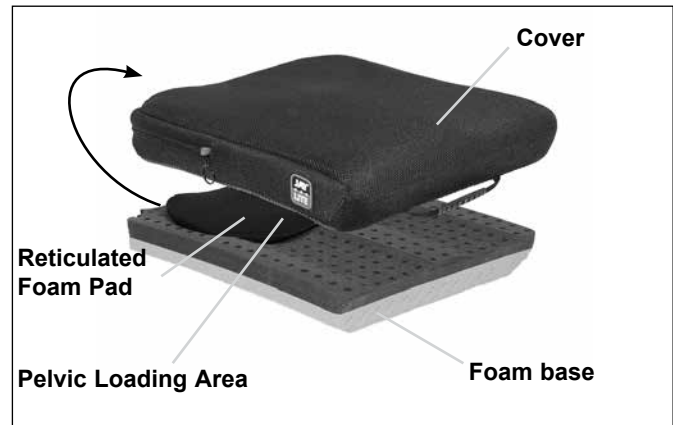
INTENDED USE

The Jay Lite is intended to provide the benefits of skin protection in a lightweight, comfortable and low-maintenance design. The Jay Lite incorporates a reticulated foam pad and an Olilex foam base with a microclimatic outer cover. The Jay Lite helps to provide pressure and shear reduction which may help reduce the incidence of decubitus ulcers. It is comfortable and doesn't restrict movement, allowing the user to change positions on the cushion.

The Jay Lite cushion can be used in any wheelchair, provided that there are the proper means of attachment (Velcro strips), as well as outside a wheelchair.

The maximum user weight for this product is 150 kg.

JAY LITE COMPONENTS

FITTING A PRESSURE DISTRIBUTING CUSHION

Select a cushion to match your hip width when in the seated position. The cushion length should extend to within 1" - 2" (2.5cm - 5cm) from the back of the knees. Place the cushion to the back of the wheelchair with the material identification tags and pelvic loading area toward the rear. When fitted correctly your hips should be to the back of the chair. Your ischials (sitting bones) should be centered in the pelvic loading area of the cushion. Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures. Adjust the footrests so your thighs rest firmly but comfortably on the cushion.

NOTE: Do not use the wheelchair's seat width as a guide in determining the appropriate cushion width. Improper cushion fit may be detrimental to the performance of a pressure distributing cushion and its support of the body.

Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. Replace stretched upholstery or purchase the Jay Adjustable Solid Seat. Avoid using overly stretched backrest upholstery. Stretched back upholstery does not provide adequate support for the body and improper positioning may result. Replace stretched backrest upholstery or purchase a back support product from Sunrise that is appropriate for you. Contact an authorized supplier for a free demonstration of these products.

EASY MAINTENANCE & CLEANING

Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended.

Check the cover for tears and excessive wear and / or any other abnormalities.

Check the foam base to ensure foam consistency. Check the pelvic loading area insert to ensure reticulated foam consistency.

To clean cover

1. Remove the cover from the foam base and turn inside out.
2. Machine wash in warm water and drip or tumble dry on low heat.
3. No ironing required.

NOTE: Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.

To refit cover after washing, fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base.

NOTE: If utilized in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.

To clean PLA (pelvic loading area) insert

1. Remove the insert from the foam base.
2. It is not necessary to remove the cover from the insert to wash it. We recommend it to be washed in the cover. Before washing, enclose the insert (with cover installed) in a mesh laundry bag to protect the Velcro hooks from the washing machine agitator and prevent tangling with other items.
3. Machine wash in warm water and drip or tumble dry on low heat.

Do not wash a damaged insert.

To clean foam base

1. Remove the cover from the foam base. Wipe lightly with a damp cloth.
2. Wipe off the base with a clean cloth. Let it completely air-dry before reassembly.
3. Reinstall the cover with the reticulated foam pad over foam base. Ensure the back of the cover matches up to the back of the foam base.

NOTE:

- Consult table on next page for further approved cleaning methods

Laundry Care Symbols

Machine wash in hot water (70° C)

Only non-chlorine bleach, when needed

Drip dry

Do not iron

Do not dry clean

NOTE: 70°C is the maximum temperature. These instructions apply to the cover only.

REASSEMBLING THE JAY LITE CUSHION AFTER CLEANING

Follow cleaning and maintenance instructions as described in "Easy Maintenance and Cleaning" then reassemble as follows:

Place the PLA insert on the foam base, fixing it to the base through the existing Velcro strips, and then introduce the base with PLA insert into the outer cover. Ensure that the zipper is at the rear of the foam base (the part number tag should be to the rear of the foam base) where the pelvic loading area is located.

JAY LITE CUSHION WARRANTY

Every Jay Lite cushion is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

There is no implied warranty beyond what is contained herein. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

This does not affect your statutory rights.

Approved Cleaning Procedures for Jay Cushions

Product	Jay Lite	Jay J3	J2	Jay Easy Visco	Jay Soft Combi	Jay Xtreme Active	Jay Basic	Jay Easy	Jay Balance
Outer Cover - Microclimatic and Stretch									
Machine Washing 70°C 									
Outer Cover - Incontinence									
Machine Washing 60°C 									
Inner Cover									
Spray and wipe disinfectant 	NA	NA	NA	NA	NA		NA	NA	
Cushion Core (foam base)									
Steam autoclave up to 105°C (1) 									
Spray and wipe disinfectant 									
Immersion disinfectant (2) 									
Cushion Elements									
Roho Air element	NA			NA	NA	NA	NA	NA	
Positioning elements	NA			NA	NA		NA	NA	
Fluid pad	NA			NA	NA		NA		
(1) An autoclave is a self locking machine that sterilizes with steam under pressure. (2) At low temperature.									
The product warranty shall not apply to products damaged due to the failure to follow these cleaning instructions or the use of inappropriate cleaning compounds.									

Recommended Disinfectants

All disinfectants which are listed and recommended by the Robert Koch Institute:
http://www.rki.de/EN/Home/homepage_node.html

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Manufactured under one or more of the following patents plus other patents pending: U.S. Patent #

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Sunrise Medical
North American Headquarters
2842 Business Park Avenue
Fresno, CA, 93727, USA
(800) 333-4000
(800) 300-7502
www.SunriseMedical.com

Sunrise Medical
Thorns Road
Brierley Hill
West Midlands
DY5 2LD
Phone: 0845 605 66 88
Fax: 0845 605 66 89
www.SunriseMedical.co.uk

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Sunrise Medical Pty. Ltd.
6 Healey Circuit, Huntingwood, NSW 2148,
Australia
Phone: 9678 6600,
Orders Fax: 9678 6655,
Admin Fax: 9831 2244.
www.SunriseMedical.au

Sunrise Medical Poland Sp. z o.o.
ul. Elektronowa 6,
94-103 Łódź
Poland
Telefon: + 48 42 275 83 38
Fax: + 48 42 209 35 23
E-mail: pl@sunrisemedical.de
Sunrise-Medical.pl

Sunrise Medical Canada Inc. -
Concord Head Office
237 Romina Drive, Unit 3
Concord, Ontario L4K 4V3
Canada
Telephone: (+1) 905 660 6704 (Administration)
Telephone 2: (+1) 888 703 9021 (Toll Free)
Fax: (+1) 905 660 2460 (Main)

©2011 Sunrise Medical (US) LLC

OM_Jay Lite_Rev.4.0_EU_ML_30_06_2014
112803 Rev. D Lite_OM