



JAY LITE

FEEL THE LITE



ULTRA LIGHTWEIGHT...

The perfect complement to a lightweight wheelchair, the JAY LITE is the lightest cushions in its class. Performing very well in terms of skin protection, stability and comfort, but weighing from just 660 grams, the JAY LITE is the ideal cushion for the active user.



... AND SUPERIORLY BREATHABLE

An active life creates heat. That's why the JAY LITE uses very breathable materials to facilitate air-flow under the user whilst in their wheelchair. For example, the 3DX™ microclimatic cover and Olotex „airflow“ base with air vents provide superior heat and moisture dissipation through the cushion's material.

Stay cool and look „cool“!





POSITIONED FOR ACTION

The JAY LITE cushion provides great skin protection, stability and comfort. To minimise the risk of pressure sores, the anthropometrically designed Optiwell pelvic loading area (PLA) effectively reduces loading on the ischial tuberosities without overloading the trochanters. Its lightweight reticulated foam insert helps to distribute load with a minimum of weight. Plus, the pre-contoured foam base provides the support you need to keep active!

Pelvic loading area


redistributes weight to reduce the risk of pressure sores

Sacral notch

protects the coccyx area

Ischial well cutouts

reduces pressure under the ischial tuberosities to protect skin integrity

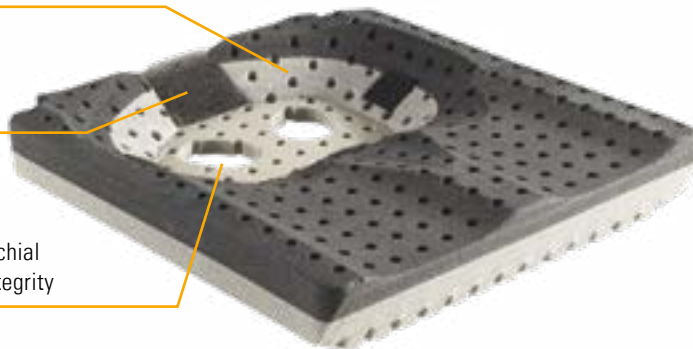


Reticulated foam insert

superior breathability, washable and exchangeable

Pre-contoured base

increased stability and leg support

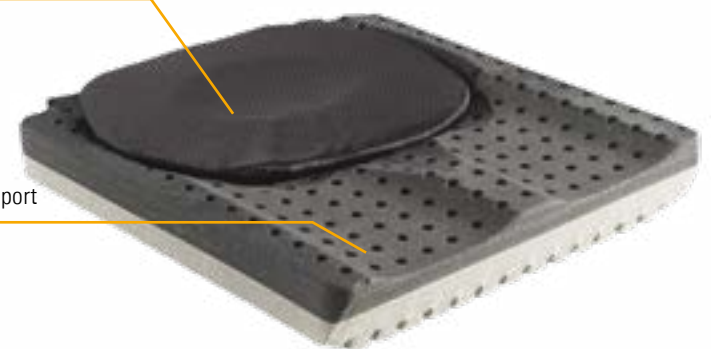



// 1 CUSHION WITH THE CHOICE OF 2 DIFFERENT BASE CONTOURS.

The JAY LITE cushions offer 2 different contoured base versions. The standard base is ideal for most users who require comfort and pressure redistribution. Its low-profile contour also allows for easier transfers in and out of the wheelchair. Alternatively, for those who require a higher level of postural stability, the JAY LITE P version provides lateral and medial thigh support which increases the overall stability of the user.

→ **JAY LITE – low profile contour**
Provides mild leg support and allows easy transfers.



→ **JAY LITE P – positioning contour**
More leg positioning through lateral and medial thigh support for increased overall stability.



A FUNCTIONALLY WELL THOUGHT-OUT DESIGN.

The JAY LITE's design reflects the needs and lifestyle of an active user. The bevelled front edge prevents rubbing against the back of the legs, eliminating the need for using a shorter length. The corner cut at the lower sides of the cushion also reduces the load on the trochanter (and reduces the seat-to-floor height). The adapted carrying handle offers an easy grip and the notches for back canes locks the cushion into place.



Gray rubber non-stick bottom
breathable and anti-slip



Corner cut
to reduce load on the trochanters



Beveled front edge
prevents rubbing against
the back of the legs

Adapted carrying handle
for easy grip

Select your cushion insert.

The JAY LITE cushion offers two different well inserts. The reticulated foam insert uses an air-flow cover and is superiorly breathable. The whole insert is also machine washable. The optional PU foam insert comes with an incontinent cover that's water-resistant and easy to wipe-off and clean.

→ **Reticulated foam insert**
with air flow cover. Superior
breathability, machine washable.



→ **PU foam insert (optional)**
with incontinent cover. Insert
cover is water-resistant and
easy to wipe off and clean.



TECHNICAL DATA

Cushion width:	35 cm – 50 cm
Cushion depth:	35 cm – 50 cm
Cushion height:	8 cm
User weight capacity:	150 kg
Cushion weight:	660 g - 1090 g
Insert:	Reticulated foam insert with air flow cover (standard) & PU foam insert with incontinent cover (optional)
Outer cover:	Microclimatic cover
Base versions:	Standard version Positioning version - contoured base for a better leg positioning



For further information on the full specification, options and accessories, please refer to the order form.

All information is subject to change without notification. Please consult Sunrise Medical with any queries you may have.

If you are visually impaired, this document can be viewed in PDF format at www.SunriseMedical.com or alternatively is available on request in large text.



Sunrise Medical GmbH & Co. KG

Kahlbachring 2-4
D-69254 Malsch/HD
Tel.: +49 (0) 72 53 / 9 80-0
Fax: +49 (0) 72 53 / 9 80-222
E-Mail: kundenservice@sunrisemedical.de
www.SunriseMedical.de