JAY Shape™ Back

Important Consumer Information

NOTE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

WARRANTY REGISTRATION: To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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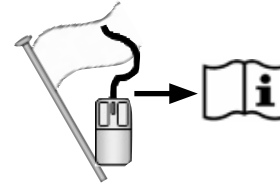
3

ENGLISH

Introduction	3
installation	4
adjustment	5
Maintenance And Cleaning	6
Recycling / re-use / Additional Information	7
Additional Information	8

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GENERAL WARNINGS/INTENDED USE

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a JAY Shape is appropriate.

WARNING!

- Backs should only be installed by:
- An authorized Sunrise Medical supplier.
- A qualified and competent healthcare professional who has undergone appropriate certified training by either sunrise medical or their approved supplier.

THE JAY SHAPE INTENDED USE

The JAY Shape is designed to provide the important benefits of a proper back support and additional comfort for clients with spinal deformities such as kyphosis (curvature of the spine). It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is light-weight, durable and easy to use.

Maximum user weight: 135 kg (297.5 lbs).
The intended lifetime of the back rest is 2 years.

WARNING!

- Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury.
- Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

WARNING!

Do not use JAY Shape to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

WARNING!

Do not use the JAY Shape cover or straps to lift wheelchair.

WARNING!

Carelessness causes fire.

WARNING!

Please read this manual before use.

JAY SHAPE COMPONENTS

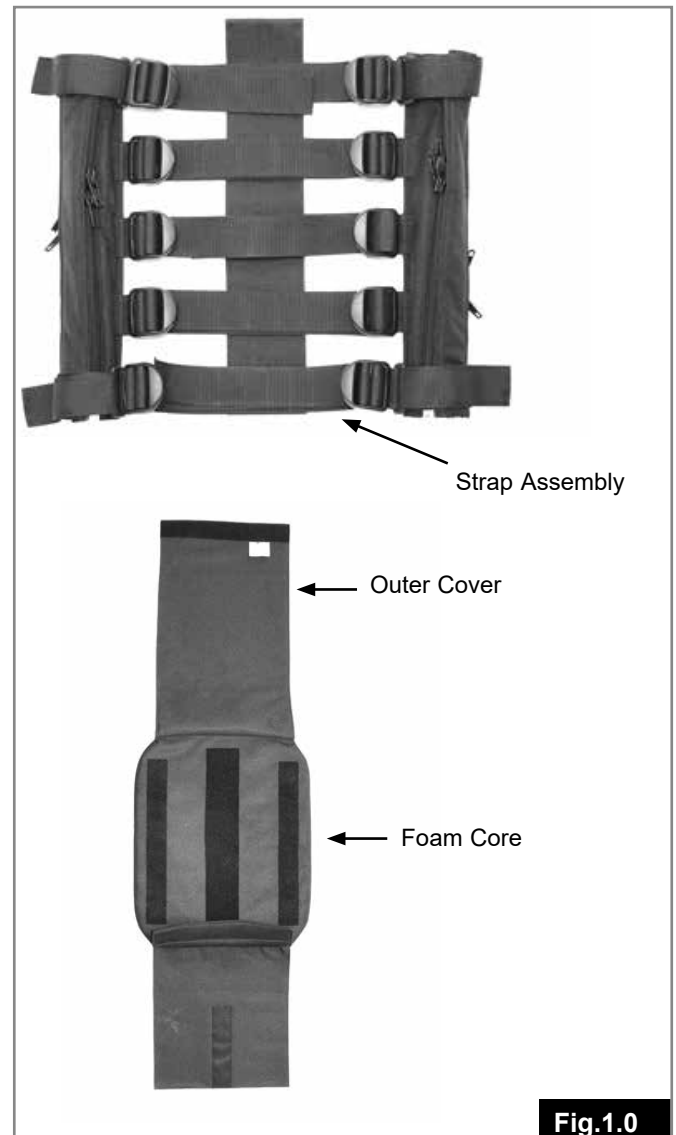
Fig.1.0

INSTALLATION

Wheelchair Types:

The JAY Shape is designed to be compatible with most wheelchairs with the following exceptions: Wheelchairs that are angularly adjustable, recline or tilt and result in a back angle greater than 60 degrees from the vertical should not be used.

WARNING!

Use with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair resulting in a fall and the potential for serious injury.

Installation:

To install the JAY Shape, first remove the outer cover from the strap assembly. Then install the strap assembly onto the back canes of the wheelchair by opening the zippers around the back canes and closing the zippers again fully, (Fig.2.0). Install the top and bottom strap around the outside of the back canes (Fig. 2.0).

CAUTION!

Note the height of the back. Make sure the strap assembly is appropriate to the top of the back canes and leaves enough room underneath the strap assembly for the cushion.

NOTE: Align grommet holes of the JAY Shape with the grommet holes (if present) on the wheelchair. If provided, install the wheelchair back cane screws as shown in, (Fig.3.0).

- Ensure the buckles of the adjustment straps are facing rearward.
- Make sure the top and bottom strap is installed around the outside of the back canes.
- Adjust each strap individually to achieve the required tension.
- Then install the cover making sure the padded side of the cover is next to the back of the client. Lastly, fold the back flap of the cover over the back of the strap assembly, (Fig.4.0).

Fig.2.0

Fig.3.0

Fig.4.0

ADJUSTMENT

To adjust the tension of the JAY Shape:

- Lift up the back cover flap to expose the horizontal straps.
- Adjust each strap individually to the desired tension.
- Fold down the back cover flap over the horizontal adjustment straps to complete your adjustment.

NOTE: You may adjust the JAY Shape while the user is seating on the chair.

CAUTION!

AVOID over-tightening of the strap assembly as this may cause damage to the integrity of the back canes.

Installing the lateral supports:

1. Determine the desired location of the lateral supports on the back tubes, (Fig.5.0).
2. Attach and align receiver on the back tubes, (Fig.5.0).
3. Repeat the process for the second lateral support if applicable.
4. Install the back cane covers and open the zippers to leave space for the lateral supports accordingly, (Fig.6.0).
5. To adjust the angle of the lateral supports, unscrew the bolts as described in the picture, (Fig.7.0).

MAINTENANCE AND CLEANING

Sunrise recommends that the Jay Shape Strap Assembly be checked for wear, such as loose, worn or broken components every 6 months. Loose straps should be retightened according to the installation instructions. Contact your authorized Sunrise Medical supplier immediately to replace any broken components. Do not continue to use the system after identifying loose, worn or broken components.

EASY CLEANING

NOTE: When disassembling the back for cleaning, note the position of the Strap Assembly and cover for correct reassembly.

To clean the outer cover:

Wipe down the outer cover using a soft cloth and warm soap and water.
The outer cover can be machine washed in warm water (40°) and light detergent as needed.

NOTE: Remove foam from inside of cover prior to machine washing.

The foam core may be washed by hand with light soap and water.

Strap Assembly

Wipe down strap assembly as needed using light soap and warm water.

 **CAUTION!**

Do not machine wash the strap assembly.
If you have any additional questions regarding cleaning procedures, please contact your authorized Sunrise Medical dealer.

Laundry Care Symbols	
	Machine wash in warm water: (40° C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	Do not dry clean

JAY SHAPE WARRANTY

Every Jay Shape is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorised supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

There is no implied warranty beyond what is contained herein. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

This does not affect your statutory rights.

RECYCLING / RE-USE:**WARNING!****Risk of Injury.**

- Always have an authorized Sunrise Medical dealer inspect the product for damage or wear prior to transferring it to another user.
- If any type of damage is found, **DO NOT USE THIS PRODUCT.**
- Failure to heed these warnings may lead to severe injury.

This Product is suitable for re-use*:

- Have an authorized Sunrise Medical dealer inspect the strap assembly, foam and cover for damage or wear.
- If any damage is found, do not re-use.
- Clean and disinfect the strap assembly, foam and cover in accordance with local government regulations.
- Refer to your local Infection Control Office for advice.
- After cleaning and disinfection, have an authorized Sunrise Medical dealer inspect the strap assembly, foam and cover to insure that the cleaning/ disinfection process did not cause any damage.

ADDITIONAL INFORMATION**Indications**

The varieties of backrest fitting variants, as well as the different positioning components to deliver postural support, pressure distribution, increase seating tolerance are benefits according to each user requirement and indication, mean that it can be used by a range of users with different positioning needs and indications as follows:

- Spinal Cord Injury
- MS
- Neurological disease pattern (hemiplegia, M. Parkinson)
- Extremity defect deformity
- Joint contracture
- Amputation
- Tetraplegia
- Cerebral Palsy
- Muscle Dystrophy

Contraindications

- No contraindications are known or available

ISO-13485 Statement

We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R&D to production. This product meets the requirements in accordance with EU regulations. Options or accessories shown are available at extra cost.

1

Type: Shape

(01) 04050063057196 (10) 12345

2

Shape

Date:

cm cm

SUNRISE MEDICAL GmbH
Kahlbachring 2-4
69254 Malsch / HD
Germany

See owner's manual Maintenance & Cleaning section for explanation of symbols

UK RP Sunrise Medical Limited
Thorns Road, Brierley Hill
West Midlands, DY5 2LD
UNITED KINGDOM

SUNRISE MEDICAL

Shape

EN Back
ES Respaldo
PT Encosto
IT Schienale
FR Dossier
DE Rücken
NL Rugleuning
DA Rygstøtte
SV Ryggstöd
NO Rygg
FI Selkätuki
CS Opěradlo
EL Πλάτη
PL Oparcie

Device Type Translation Label

EN			
1 Type:	Product Name/ SKU Number	XXXX-XX-XX	Date of Manufacture
2 Date:	LOT: defined by Julian Date	Part Code	Assembly Part Number
	Back width	MD	This symbol means Medical Device
	Back height		Manufacturer's address
	Max user weight XXX kg	EC REP	European Authorised Representative
CE	CE Mark	UK RP	UK Responsible Person
UK CA	UKCA Mark	CH REP	Swiss Representative's address
	Consult instructions for use		Importer's address

The management system of SUNRISE MEDICAL is certified to EN ISO 13485 and ISO 14001.

We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R & D to production. This product meets the requirements in accordance with EU and UK regulations. Options or accessories shown are available at extra cost.

The varieties of backrest fitting variants, as well as the different positioning components to deliver postural support, pressure distribution, increase seating tolerance are benefits according to each user requirement and indication, mean that it can be used by a range of users with different positioning needs and indications as follows:

- Spinal Cord Injury
- Amputation
- Joint contracture
- MS
- Tetraplegia
- Cerebral Palsy
- Neurological disease pattern (hemiplegia, M. Parkinson)
- Muscle Dystrophy
- Extremity defect deformity

No contraindications are known or available.

NOTE: General user advice. Not following these instructions may result in physical injury, damage to the product or damage to the environment!

Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

CE As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

UK CA As the manufacturer, SUNRISE MEDICAL, declares that the product conforms to the UK Medical Devices Regulation 2002 No. 618.

B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used. Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.SunriseMedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, Annex I Nr. 14.1 of the Medical Device Regulation 2017/745.

Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk.

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As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

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Sunrise Medical Australia
11 Daniel Street
Wetherill Park NSW 2164
Australia
Ph: +61 2 9678 6600
Email: enquiries@sunrisemedical.com.au
www.SunriseMedical.com.au

Sunrise Medical
North American Headquarters
2842 Business Park Avenue
Fresno, CA, 93727, USA
(800) 333-4000
(800) 300-7502
www.SunriseMedical.com

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The management system of SUNRISE
MEDICAL is certified to EN ISO 13485
and ISO 14001.

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical SAS