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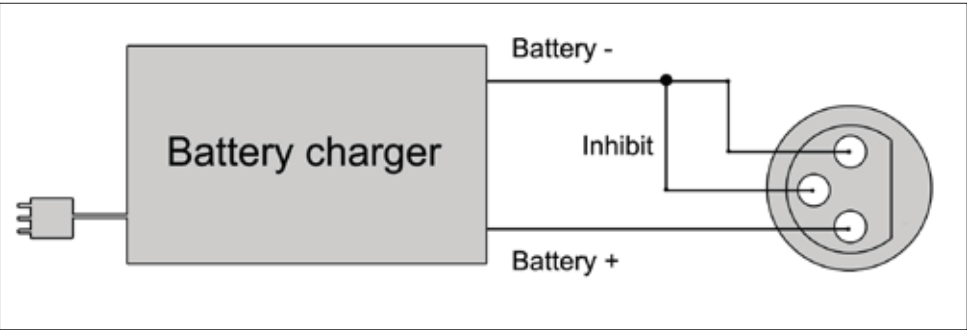
© 2016 Sunrise Medical

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of Sunrise Medical.

The information provided is based on general data concerning the construction known at the time of the publication of this manual. Sunrise Medical executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Sunrise Medical can therefore not be held liable for any damage resulting from specifications of the product deviating from the standard configuration. Illustrations contained in the manual may deviate from the configuration of your product.

The information made available has been prepared with all possible diligence, but Sunrise Medical cannot be held liable for any errors contained in the information or the consequences thereof.

Sunrise Medical accepts no liability for loss resulting from work executed by third parties.

Names, trade names, trademarks etc. used by Sunrise Medical may not, as per the legislation concerning the protection of trade names, be considered as being available.

Table of contents

1 Introduction..... 19

1.1 This user manual 19

1.2 For further information 19

1.3 Symbols used in this manual 19

2. The controller 20

2.1 DX Rem 48 controller 20

3 Driving the wheelchair with the controller 22

3.1 Switching the controller on or off 22

3.2 Driving the wheelchair 22

3.3 Speed 22

4 Operating the electrical adjustment options 23

4.1 Selecting the desired adjustment option 23

5 Lights..... 24

6. Troubleshooting..... 25

7. Locking the controller 26

8 Technical specifications..... 27

9. Technical diagrams..... 28

9.1 Technical diagram 28

9.2 Technical diagram battery charger 29

1 Introduction

1.1 This user manual

This user manual will help you to use and maintain the controller of your power wheelchair safely. This user manual is a supplement to Sunrise Medical's general wheelchair user manual. When necessary this user manual refers to other manuals as shown below:

-  Wheelchair: Refers to the general wheelchair user manual.
-  Battery charger: Refers to the user manual for the battery charger.

Read this user manual and the other user manuals referred to carefully before using the product. If one of the user manuals was not included with your wheelchair, please contact your dealer immediately. In addition to this user manual, there is also a service manual for qualified specialists.

If you are visually impaired, this document can be viewed in PDF format at www.SunriseMedical.eu or alternatively is available on request in large text.

1.2 For further information

Please contact your local, authorised Sunrise Medical dealer if you have any questions regarding the use, maintenance or safety of your wheelchair. In case there is no authorised dealer in your area or you have any questions, contact Sunrise Medical either in writing or by telephone.

For information about product safety notices and product recalls, go to www.SunriseMedical.eu

1.3 Symbols used in this manual

Note!

Pointing out possible problems to the user.

⚠ Caution!

Advice for the user to prevent damage to the product.

⚠ Warning!

Warnings for the user to prevent personal injury.

Not following these instructions may result in physical injury, damage to the product or damage to the environment!

2. The controller

2.1 DX Rem 48 controller

A controller will usually have three basic functions:

- Driving and steering a wheelchair
- Operating electrical seat adjustments
- Charging the wheelchairs batteries

There are many different control systems for wheelchairs on the market. If the controller on your wheelchair does not resemble the one in figure 1 contact your dealer.

DX Rem 48 by Dynamic Controls is a collective name for the entire control system of the wheelchair. The wheelchair is operated by means of a controller, which includes the following components:

Figure 1

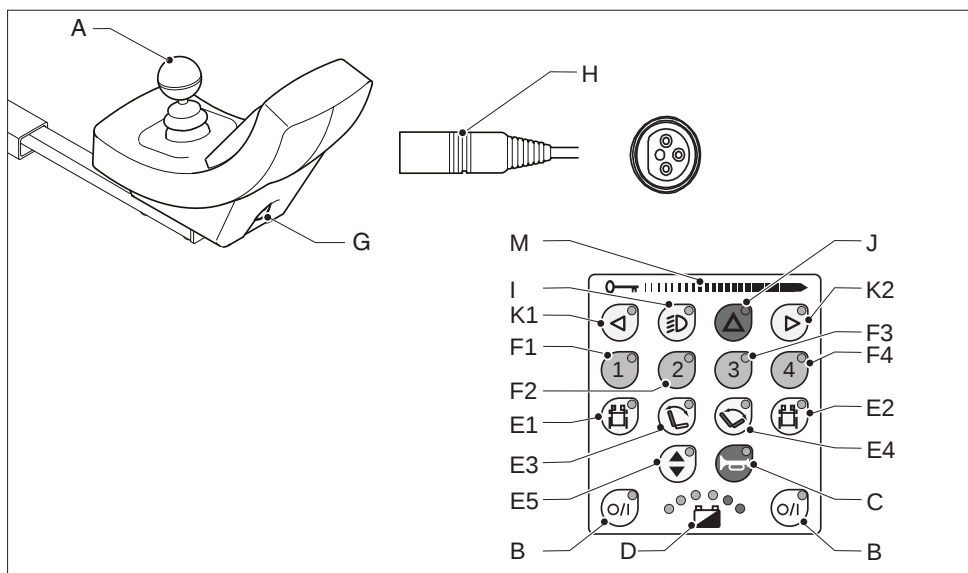
Figure 2: DX Rem 48

Part	Function
A. Joystick	In the 'drive' mode: driving and steering In the 'adjustment options' mode: • Left/right to select the adjustment options • Front/back to control the adjustment mode
B. On/off switch	Switching the controller on or off
C. Horn	Warning signal with sound
D. Battery indicator	Displays the power level of the battery
E.1 Adjustment option	Electrical adjustment of the left legrest (option)
E.2 Adjustment option	Electrical adjustment of the right legrest (option)
E.3 Adjustment option	Electrical adjustment of the angle of the backrest (option)
E.4 Adjustment option	Electrical tilt adjustment of the seat (option)
E.5 Adjustment option	High/low seat adjustment (option)
F. 1 Speed regulator	Reduce driving speed (slowest)
F. 2 Speed regulator	Increase driving speed (faster)
F. 3 Speed regulator	Increase driving speed (faster)
F. 4 Speed regulator	Increase driving speed (fastest)
G. Charge connector	Input for the battery charger
H. Charge plug for the battery charger	Connector for the battery charger
I. Lights button	Switching the lights on or off
J. Hazard lights	Warning signal with lights
K.1 Direction indicator left	Switches the left direction indicator on or off
K.2 Direction indicator right	Switches the right direction indicator on or off
M. Panel pad	Activate / deactivate the wheelchair by moving the magnetic key from left to right across the panel

3 Driving the wheelchair with the controller

3.1 Switching the controller on or off

To be able to drive or operate the electric adjustment options of the wheelchair, the controller must be switched on. Press the on/off button (B in figure 2).

3.2 Driving the wheelchair

Driving an electric wheelchair is done by operating a joystick. Move the joystick forwards and the wheelchair will also move forwards. Steer left and right and the wheelchair will turn.

3.3 Speed

The maximum speed can be controlled by the speed regulator on the controller (F 1, 2, 3 and 4 in figure 2). Speed can be controlled with the joystick while driving. If the joystick is moved a little, the wheelchair will move more slowly.

4 Operating the electrical adjustment options

Not every wheelchair has been equipped with electric adjustment options. We make a distinction between four different adjustment options to the seating system:

- Tilt adjustments
- High/low adjustments
- Backrest adjustments
- Legrest adjustments

Figure 3

4.1 Selecting the desired adjustment option

1. Switch on the controller.
2. Press on the desired adjustment option (E in figure 2). The controller will now be in the 'adjustment options' mode. The LED light on the button indicates what electrical adjustment option is enabled (1, 2, 3, 4 or 5).
3. Moving the joystick forwards and/or backwards will activate the selected electric adjustment option (see table 4). Move the joystick forwards or backwards until the desired adjustment option has been attained.

	Move joystick backwards	Move joystick forwards
Tilt adjustment	The entire chair will tilt backwards	The entire chair will tilt forwards
High/low adjustments	The entire chair will be raised	The entire chair will be lowered
Backrest adjustments	The backrest will tilt backwards	The backrest will tilt forwards
Legrest adjustments	The legrest angle will increase, the footplate will be raised	The legrest angle will decrease, the footplate will be lowered

Table 4

4. To return to the 'drive' mode: Press the selected adjustment option button again. The LED light of the speed button (F in figure 2) will light up. You are now to the 'drive' mode again.

Note: If you adjust the seat height by using the high/low option, the speed will be reduced due to safety reasons.

5 Lights

With the DX Rem 48 it is possible to control the following lights:

- Lights (I in figure 2)
- Hazard Lights (J in figure 2)
- Direction indicators (K1 & K2 in figure 2)

6. Troubleshooting

If the wheelchair will not function while the batteries are fully charged, check the following points before consulting your dealer:

- Switch the controller off and then switch it on again. Check to see if the malfunction has been solved.
- Check if the free wheel switch was switched to Drive.
- Check if the joystick was in the 0 position when the controller was switched on. In other words, the joystick must not be moved when the controller is being switched on or off.

Malfunctions list

An extended malfunction list can be found in the appendix of the service manual (for qualified specialists only). The service manual can be found on www.SunriseMedical.eu

7. Locking the controller

The Key Lock system uses a magnetic key to power the wheelchair down in such a way to prevent subsequent unauthorized power up and driving.

To arm the locking system, wipe the magnetic key from left to right on the path next to the key symbol on the DX Remote front panel. The system will beep and automatically power itself down and no LEDs should be lit.

To disarm the lock, switch on the remote and wipe the magnetic key from left to right on the path next to the key symbol on the DX Remote front panel. This will cause the LED's to stop flashing and the chair may now be driven normally. If the wheelchair is not disarmed within one minute, the remote will automatically turn itself off.

8 Technical specifications

Operating voltage range:	18V - 32V DC
Charger rating:	12A RMS Continuous, limited by DXBUS Rating
Quiescent Current:	< 1mA Off, typically 100mA On
Operating forces	Less than 800 grams
Protection rating:	IPx4
Operating Temperature Range:	-25°C to +50°C
Storage Temperature Range:	-40°C to +65°C
Operating Humidity Range:	0 to 90% RH

9. Technical diagrams

9.1 Technical diagram

The technical diagrams can also be found on the cover of the specific electronic component.

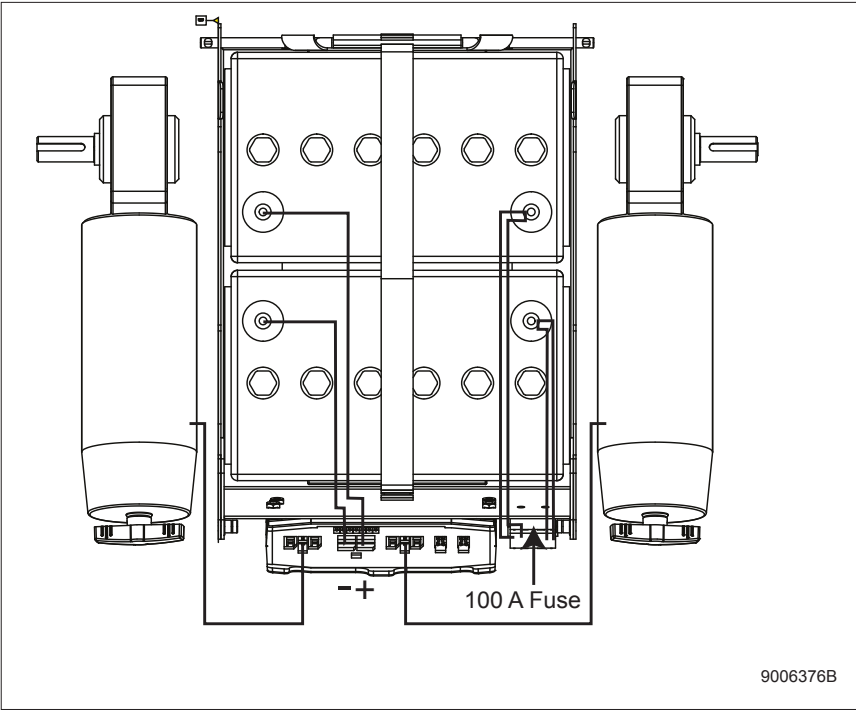
Figure 5

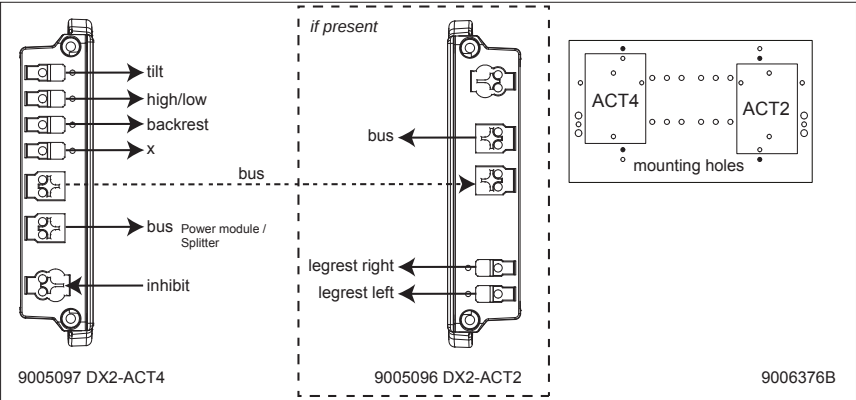
Figure 6

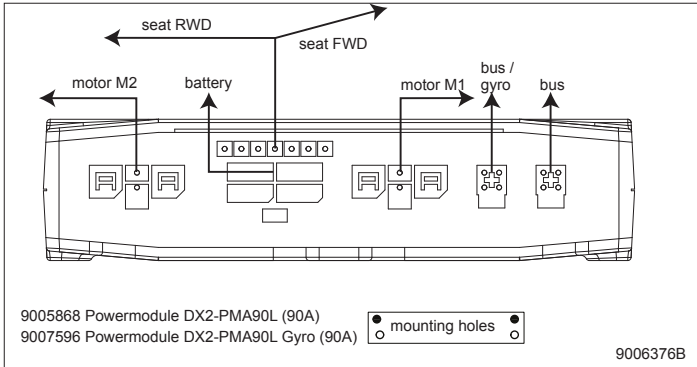
Figure 7

9.2 Technical diagram battery charger

The controller's standard configuration includes a '3-pin connection'. Ensure that the battery charger is properly connected so that the 'negative pole' and the 'inhibit' are connected, enabling the system to prevent the wheelchair from moving when the battery is being charged.

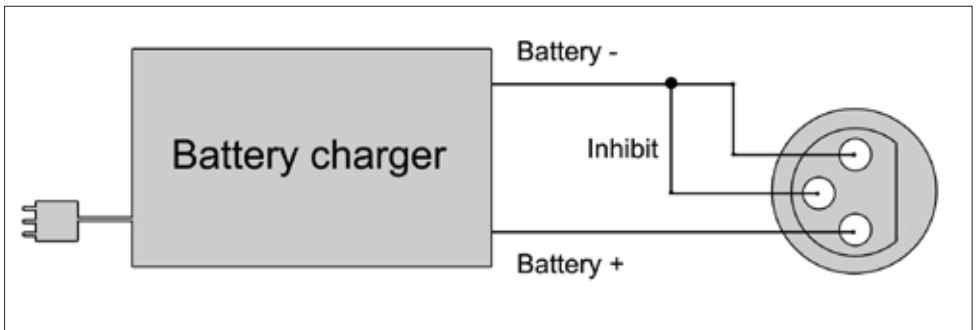
Figure 8

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Sunrise Medical AB