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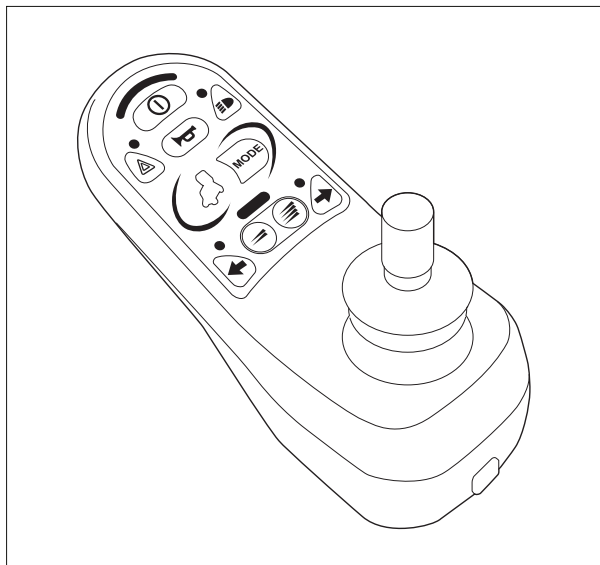
## 8 Technische specificaties

<b>Supply Voltage:</b>	24 Vdc
<b>Operating Voltage:</b>	16Vdc to 35Vdc
<b>Peak Voltage:</b>	35Vdc
<b>Reverse Battery Current:</b>	-40Vdc
<b>PWM Frequency:</b>	20kHz $\pm$ 0.5%
<b>Brake Voltage:</b>	12/24Vdc
<b>Brake Current:</b>	200 $\mu$ A min. 1A max.
<b>Charger Connector:</b>	Use only Neutrik NC3MX
<b>Battery Charging Current:</b>	12Arms max.
<b>Maximum Drive Current:</b>	R-net 80 80A R-net 90 90A R-net 120 120A
<b>Indicator Outputs:</b>	45W per side
<b>Lighting Outputs:</b>	21W per side
<b>Brake Light Output:</b>	42W total
<b>Actuator Current:</b>	3-15A max at reduced speed (programmable). 12A max at full speed.
<b>Moisture Resistance:</b>	Electronics to IPX4
<b>Operating Temperature:</b>	Non LCD Modules -25°C to +50°C Modules with LCD Screens -10°C to +50°C
<b>Storage Temperature:</b>	Non LCD Modules -40°C to +65°C Modules with LCD Screens -20°C to +65°C

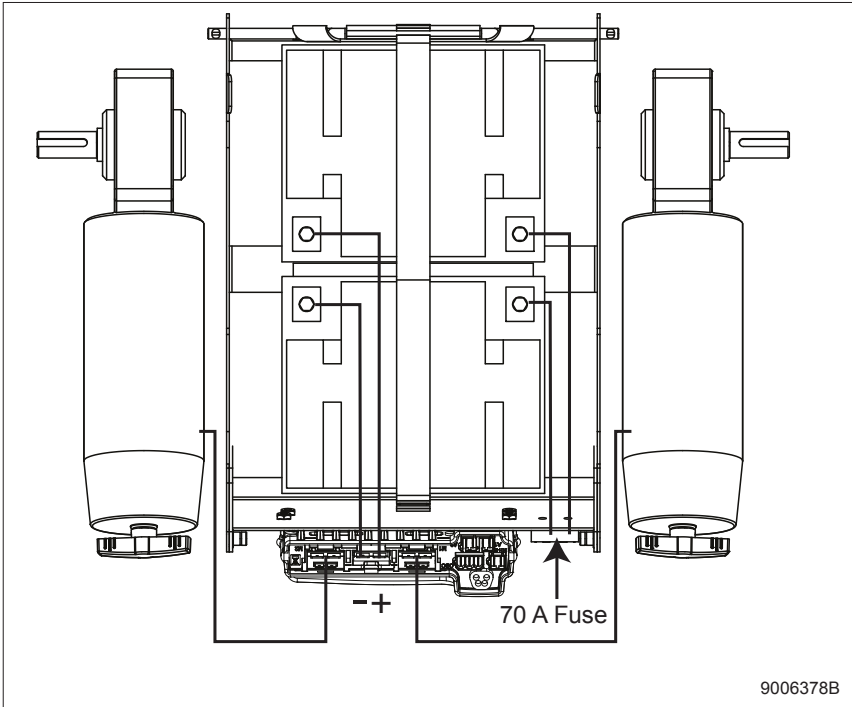
### EMC tested on sample wheelchair:

<b>Susceptibility:</b>	Tested at 30V/m to EN12184 (1999) and ANSI/ RESNA requirements
<b>Emissions:</b>	To EN55022 Class B
<b>ESD:</b>	IEC801 part 2

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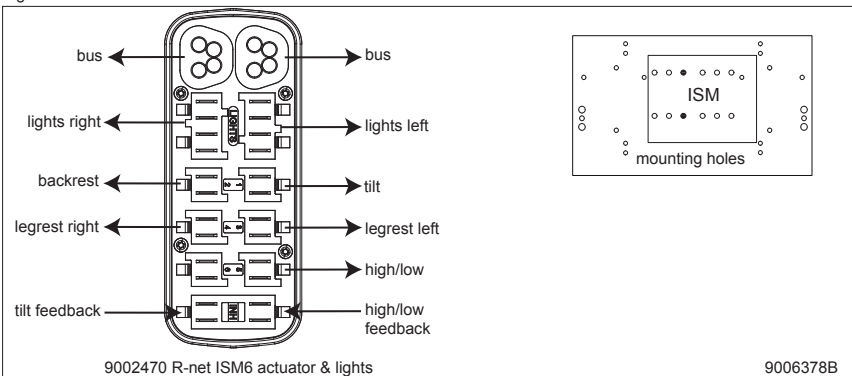
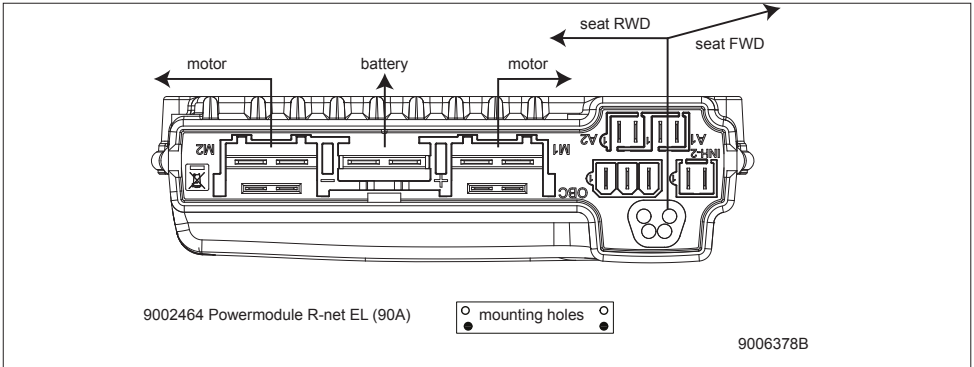
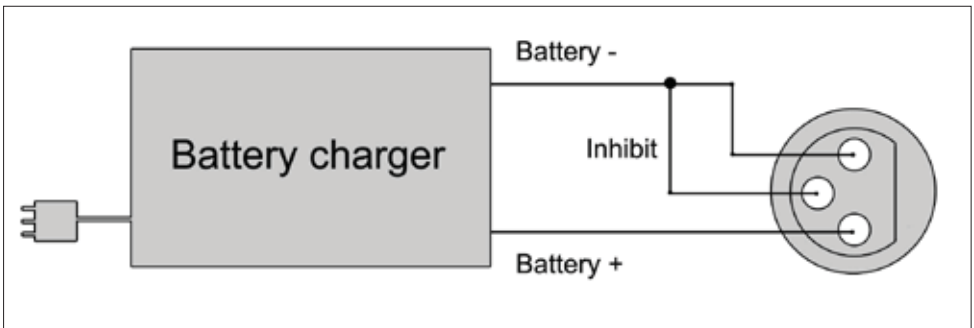
© 2016 Sunrise Medical

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of Sunrise Medical.

The information provided is based on general data concerning the construction known at the time of the publication of this manual. Sunrise Medical executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Sunrise Medical can therefore not be held liable for any damage resulting from specifications of the product deviating from the standard configuration. Illustrations contained in the manual may deviate from the configuration of your product.

The information made available has been prepared with all possible diligence, but Sunrise Medical cannot be held liable for any errors contained in the information or the consequences thereof.

Sunrise Medical accepts no liability for loss resulting from work executed by third parties.

Names, trade names, trademarks etc. used by Sunrise Medical may not, as per the legislation concerning the protection of trade names, be considered as being available.

## Table of contents

1	Introduction.....	19
1.1	This user manual	19
1.2	For further information	19
1.3	Symbols used in this manual	19
2	The controller .....	20
2.1	R-net entry level controller	20
3	Driving the wheelchair with the controller.....	22
3.1	Switching the controller on or off	22
3.2	Driving the wheelchair	22
3.3	Speed	22
4	Operating the electrical adjustment options .....	23
4.1	Selecting the desired adjustment option	23
5	Lights.....	24
6	Troubleshooting.....	25
7	Locking the controller .....	26
7.1	Keypad Locking	26
7.2	Key Locking (optional)	26
8	Technical specifications.....	27
9.	Technical diagrams.....	28
9.1	Technical diagram	28
9.2	Technical diagram battery charger	29

# 1 Introduction

## 1.1 This user manual

This user manual will help you to use and maintain the controller of your power wheelchair safely. This user manual is a supplement to Sunrise Medical's general wheelchair user manual. When necessary this user manual refers to other manuals as shown below:

-  Wheelchair: Refers to the general wheelchair user manual.
-  Battery charger: Refers to the user manual for the battery charger.

**Read this user manual and the other user manuals referred to carefully before using the product.** If one of the user manuals was not included with your wheelchair, please contact your dealer immediately. In addition to this user manual, there is also a service manual for qualified specialists.

**If you are visually impaired, this document can be viewed in PDF format at [www.SunriseMedical.eu](http://www.SunriseMedical.eu) or alternatively is available on request in large text.**

## 1.2 For further information

Please contact your local, authorised Sunrise Medical dealer if you have any questions regarding the use, maintenance or safety of your wheelchair. In case there is no authorised dealer in your area or you have any questions, contact Sunrise Medical either in writing or by telephone.

For information about product safety notices and product recalls, go to [www.SunriseMedical.eu](http://www.SunriseMedical.eu)

## 1.3 Symbols used in this manual

### Note!

Pointing out possible problems to the user.

### ⚠ Caution!

Advice for the user to prevent damage to the product.

### ⚠ Warning!

Warnings for the user to prevent personal injury.

Not following these instructions may result in physical injury, damage to the product or damage to the environment!

## 2 The controller

### 2.1 R-net entry level controller

A controller will usually have three basic functions:

- Driving and steering a wheelchair
- Operating electrical seat adjustments
- Charging the wheelchairs batteries

There are many different control systems for wheelchairs on the market. If the controller on your wheelchair does not resemble the one in figure 1, contact your dealer.

R-net entry level by PG Driving Technologies is a collective name for the entire control system of the wheelchair. The wheelchair is operated by means of a controller, which includes the following components:

Figure 1

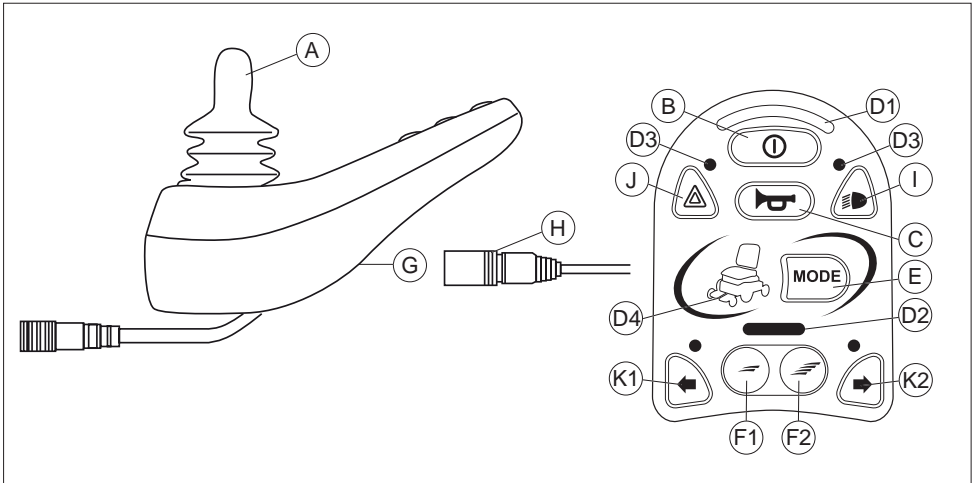
Figure 2

Part	Function
A. Joystick	In the 'drive' mode: driving and steering In the 'adjustment options' mode: <ul style="list-style-type: none"> <li>• Left/right to select the adjustment options</li> <li>• Front/back to control the adjustment options</li> </ul>
B. On/off switch	Switching the controller on or off
C. Horn	Warning signal with sound
D.1 Battery indicator	Displays the power level of the battery
D.2 Maximum speed	Displays the maximum speed limit as set by the user
D.3 Light and/or indicator	Displays the status of lights and/or indicator
D.4 Adjustment option	Displays the currently selected electrical seating adjustment
E. "Mode" button	To select the electrical seating adjustment
F.1 Speed regulator	Reduce driving speed (slower)
F.2 Speed regulator	Increase driving speed (faster)
G. Charge connector	Input for the battery charger
H. Charge plug for the battery charger	Connector for the battery charger
I. Lights button	Switching the lights on or off
J. Hazard lights	Warning signal with lights
K1: Direction indicator left	Switches the left direction indicator on or off
K2: Direction indicator right	Switches the right direction indicator on or off

## **3 Driving the wheelchair with the controller**

### **3.1 Switching the controller on or off**

To be able to drive or operate the electronic adjustment options of the wheelchair, the controller must be switched on. Press the on/off button (B in figure 2).

### **3.2 Driving the wheelchair**

Driving an electric wheelchair is done by operating a joystick. Move the joystick forwards and the wheelchair will also move forwards. Steer left and right and the wheelchair will turn.

### **3.3 Speed**

The maximum speed can be controlled by the speed regulator on the controller (F in figure 2). The speed bar will display the maximum speed (D2 in figure 2). Speed can be controlled with the joystick while driving. If the joystick is moved a little, the wheelchair will move more slowly.

## 4 Operating the electrical adjustment options

Not every wheelchair has been equipped with electrical adjustment options. We make a distinction between four different adjustment options to the seating system:

- Tilt adjustments
- High/low adjustments
- Backrest adjustments
- Legrest adjustments

### 4.1 Selecting the desired adjustment option

1. Switch on the controller
2. Press on the 'mode' button to select the 'adjustment options' mode (E in figure 2). The controller will now be in the 'adjustment options' mode. It is also possible to follow the status.

The joystick is used to select and operate the electric adjustment options.

3. Move the joystick to the left or the right to select the desired adjustment option. The actuator LED's indicates the currently selected seating function.
4. Moving the joystick forwards and/or backwards will activate the selected electric adjustment option (see table 3). Move the joystick forwards or backwards until the desired adjustment option has been attained.

	<b>Move joystick backwards</b>	<b>Move joystick forwards</b>
<b>Tilt adjustment</b>	The entire chair will tilt backwards	The entire chair will tilt forwards
<b>High/low adjustments</b>	The entire chair will be raised	The entire chair will be lowered
<b>Backrest adjustments</b>	The backrest will tilt backwards	The backrest will tilt forwards
<b>Legrest adjustments</b>	The legrest angle will increase, the footplate will be raised	The legrest angle will decrease, the footplate will be lowered

Table 3 Electrical adjustments

5. To return to the 'drive' mode: Press on the 'mode' button to select the 'drive' mode.

*Note: If you adjust the seat height by using the high/low option, the speed will be reduced due to safety reasons.*

## 5 Lights

With the R-net entry level it is possible to control the following lights:

- Lights (I in figure 2)
- Hazard Lights (J in figure 2)
- Direction indicators (K1 & K2 in figure 2)

## 6. Troubleshooting

If the wheelchair will not function while the batteries are fully charged, check the following points before consulting your dealer:

- Switch the controller off and then switch it on again. Check to see if the malfunction has been solved.
- Check if the free wheel switch was switched to Drive.
- Check if the joystick was in the 0 position when the controller was switched on. In other words, the joystick must not be moved when the controller is being switched on or off.

### **Malfunctions list**

An extended malfunction list can be found in the appendix of the service manual (for qualified specialists only). The service manual can be found on [www.SunriseMedical.eu](http://www.SunriseMedical.eu)

## 7 Locking the controller

The control system can be locked in one of two ways. Either using a button sequence on the keypad or with a physical key. How the control system is locked depends on how the wheelchair manufacturer has programmed the system.

### 7.1 Keypad Locking

To lock the wheelchair using the keypad;

- While the control system is switched on, depress and hold the on/off button.
- After 1 second the control system will beep. Now release the on/off button
- Deflect the joystick forwards until the control system beeps.
- Deflect the joystick in reverse until the control system beeps.
- Release the joystick, there will be a long beep.
- The wheelchair is now locked.

To unlock the wheelchair using the keypad;

- If the control system has switched off, press the on/off button.
- Deflect the joystick forwards until the control system beeps.
- Deflect the joystick in reverse until the control system beeps.
- Release the joystick, there will be a long beep.
- The wheelchair is now unlocked.

### 7.2 Key Locking (optional)

To lock the wheelchair with a key;

- If the control system has switched on.
- Insert and remove a PGDT supplied key into the charger socket on the joystick module.
- A short beep will be heard.
- The wheelchair is now locked.

To unlock the wheelchair;

- If the control system has switched off, press the on/off button.
- Insert and remove a PGDT supplied key into the charger socket on the joystick module.
- A short beep will be heard.
- The wheelchair is now unlocked.

## 8 Technical specifications

<b>Supply Voltage:</b>	24 Vdc
<b>Operating Voltage:</b>	16 Vdc to 35 Vdc
<b>Peak Voltage:</b>	35 Vdc
<b>Reverse Battery Current:</b>	-40 Vdc
<b>PWM Frequency:</b>	20 kHz $\pm$ 0.5 %
<b>Brake Voltage:</b>	12/24 Vdc
<b>Brake Current:</b>	200 $\mu$ A min. 1A max.
<b>Charger Connector:</b>	Use only Neutrik NC3MX
<b>Battery Charging Current:</b>	12Arms max.
<b>Maximum Drive Current:</b>	R-net 80 80A R-net 90 90A R-net 120 120A
<b>Indicator Outputs:</b>	45W per side
<b>Lighting Outputs:</b>	21W per side
<b>Brake Light Output:</b>	42W total
<b>Actuator Current:</b>	3-15A max at reduced speed (programmable). 12A max at full speed.
<b>Moisture Resistance:</b>	Electronics to IPX4
<b>Operating Temperature:</b>	Non LCD Modules -25°C to +50°C Modules with LCD Screens -10°C to +50°C
<b>Storage Temperature:</b>	Non LCD Modules -40°C to +65°C Modules with LCD Screens -20°C to +65°C

### EMC tested on sample wheelchair:

<b>Susceptibility:</b>	Tested at 30V/m to EN12184 (1999) and ANSI/ RESNA requirements
<b>Emissions:</b>	To EN55022 Class B
<b>ESD:</b>	IEC801 part 2

## 9. Technical diagrams

### 9.1 Technical diagram

The technical diagrams can also be found on the cover of the specific electronic component.

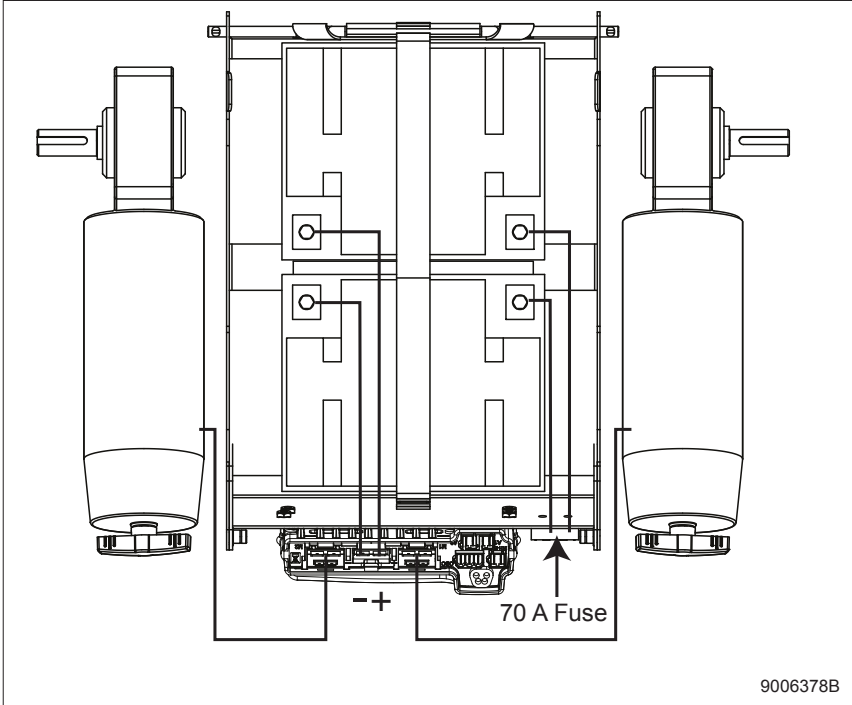
Figure 4

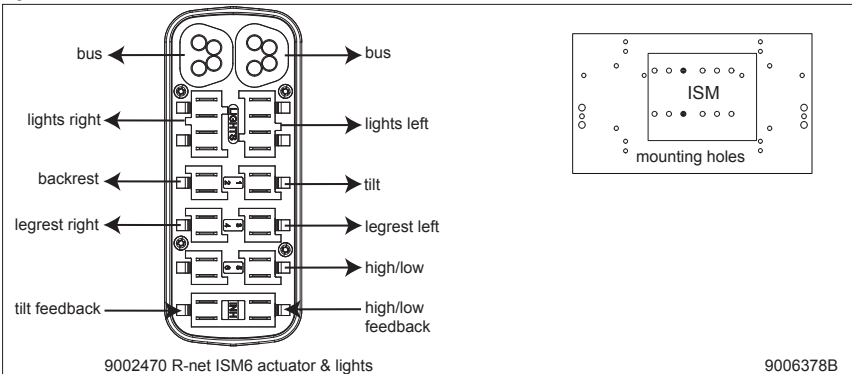
Figure 5

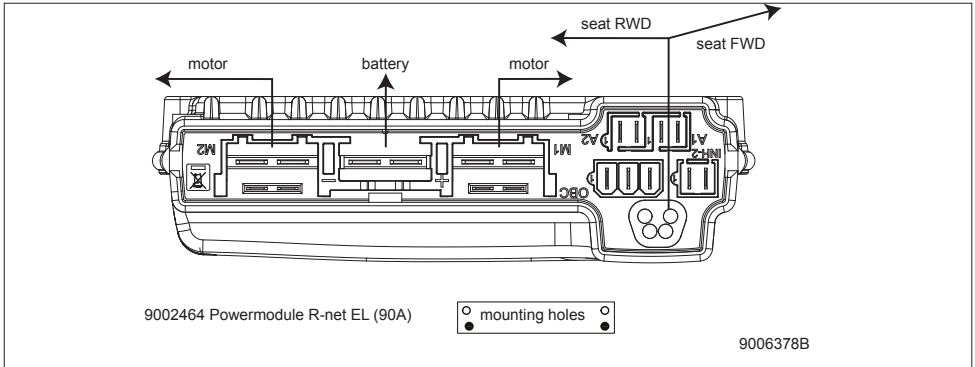
Figure 6

## 9.2 Technical diagram battery charger

The controller's standard configuration includes a '3-pin connection'. Ensure that the battery charger is properly connected so that the 'negative pole' and the 'inhibit' are connected, enabling the system to prevent the wheelchair from moving when the battery is being charged.

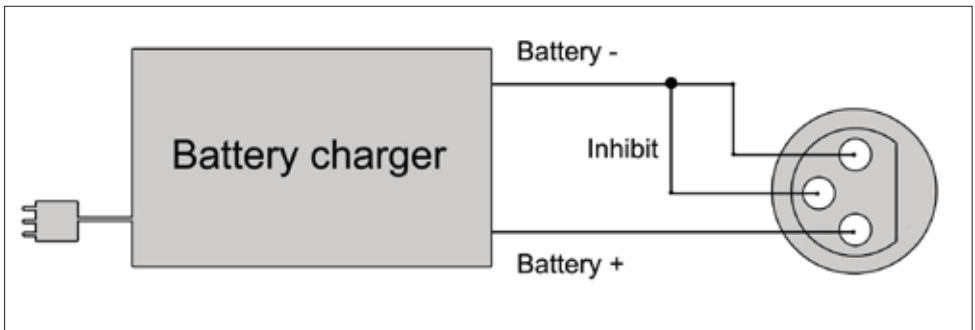
Figure 7

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Sunrise Medical AB