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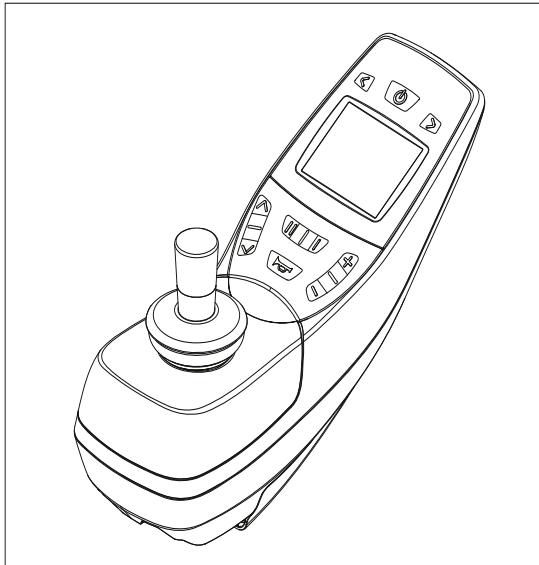
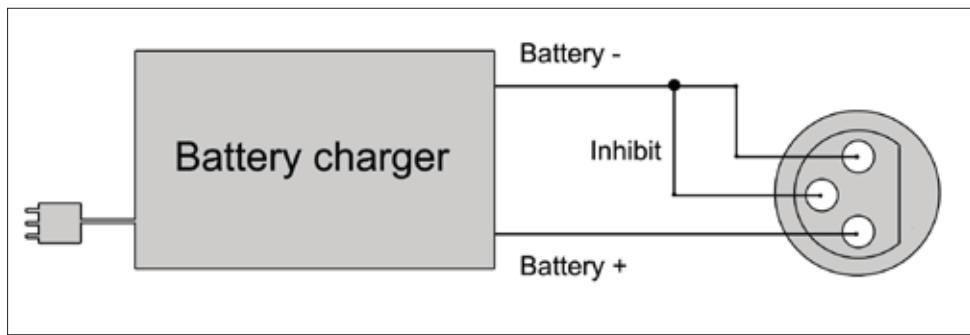
© 2016 Sunrise Medical

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of Sunrise Medical.

The information provided is based on general data concerning the construction known at the time of the publication of this manual. Sunrise Medical executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Sunrise Medical can therefore not be held liable for any damage resulting from specifications of the product deviating from the standard configuration. Illustrations contained in the manual may deviate from the configuration of your product.

The information made available has been prepared with all possible diligence, but Sunrise Medical cannot be held liable for any errors contained in the information or the consequences thereof.

Sunrise Medical accepts no liability for loss resulting from work executed by third parties.

Names, trade names, trademarks etc. used by Sunrise Medical may not, as per the legislation concerning the protection of trade names, be considered as being available.

Table of contents

1	Introduction.....	21
1.1	This user manual	21
1.2	For further information	21
1.3	Symbols used in this manual	21
2.	The controller	22
2.1	DX2 REM 550	22
2.2	Display	24
2.3	Accessories	25
2.4	Functions	25
3	Driving the wheelchair with the controller.....	26
3.1	Switching the controller on or off	26
3.2	Driving the wheelchair	26
3.3	Speed	26
3.4	Driving Profile	26
3.5	Selecting a profile	26
4	Operating the electrical adjustment options	27
4.1	Selecting the desired adjustment option	27
5	Lights.....	28
6.	Troubleshooting.....	29
7.	Locking the controller	30
8	Technical specifications.....	31
9.	Technical diagrams.....	32
9.1	Technical diagram	32
9.2	Technical diagram battery charger	33

1 Introduction

1.1 This user manual

This user manual will help you to use and maintain the controller of your power wheelchair safely. This user manual is a supplement to Sunrise Medical's general wheelchair user manual. When necessary this user manual refers to other manuals as shown below:

-  Wheelchair: Refers to the general wheelchair user manual.
-  Battery charger: Refers to the user manual for the battery charger.

Read this user manual and the other user manuals referred to carefully before using the product. If one of the user manuals was not included with your wheelchair, please contact your dealer immediately.

In addition to this user manual, there is also a service manual for qualified specialists.

If you are visually impaired, this document can be viewed in PDF format at www.SunriseMedical.eu or alternatively is available on request in large text.

1.2 For further information

Please contact your local, authorised Sunrise Medical dealer if you have any questions regarding the use, maintenance or safety of your wheelchair. In case there is no authorised dealer in your area or you have any questions, contact Sunrise Medical either in writing or by telephone.

For information about product safety notices and product recalls, go to www.SunriseMedical.eu

1.3 Symbols used in this manual

Note!

Pointing out possible problems to the user.

⚠ Caution!

Advice for the user to prevent damage to the product.

⚠ Warning!

Warnings for the user to prevent personal injury.

Not following these instructions may result in physical injury, damage to the product or damage to the environment!

2. The controller

2.1 DX2 REM 550

A controller will usually have three basic functions:

- Driving and steering a wheelchair
- Operating electrical seat adjustments
- Charging the wheelchairs batteries

There are many different control systems for wheelchairs on the market. If the controller on your wheelchair does not resemble the one in figure 1 contact your dealer.

DX2 by Dynamic Controls is a collective name for the entire control system of the wheelchair. The wheelchair is operated by means of a remote controller, which includes the following components:

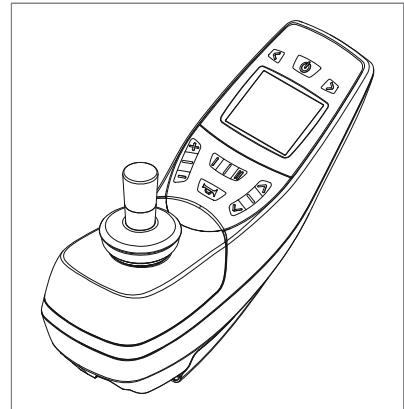
Figure 1

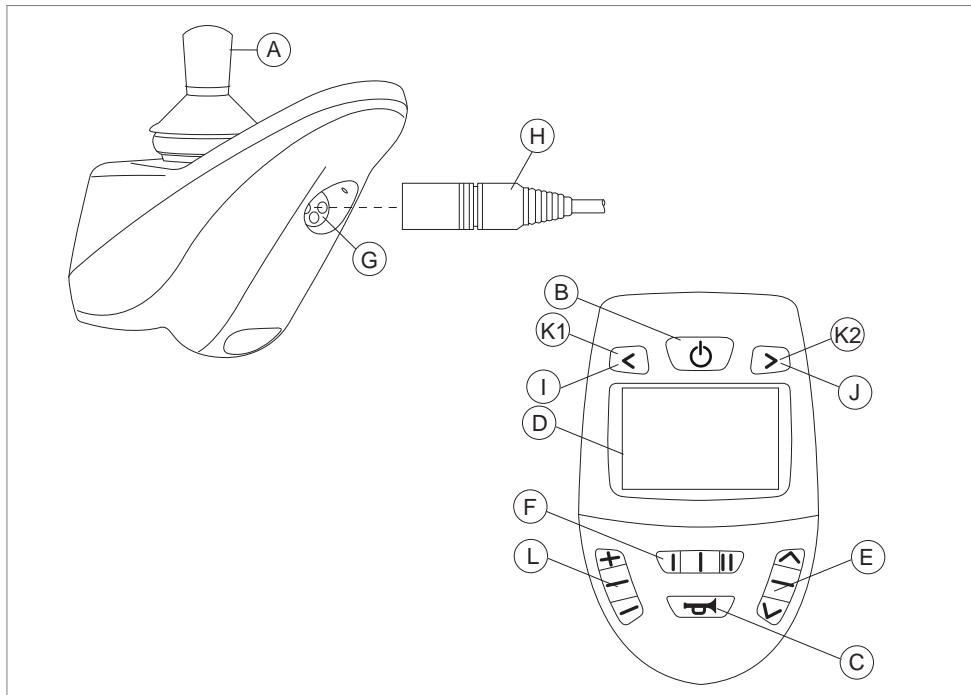
Figure 2

Part	Function
A. Joystick	In the 'drive' mode: driving and steering (to select 'drive' mode, push button L) In the 'adjustment options' mode (to select 'adjustment option mode' push button E): <ul style="list-style-type: none"> • Left/right to select the adjustment options • Front/back to control the adjustment mode
B. On/off switch	Switching the controller on or off
C. Horn	Warning signal with sound
D. LCD colour display screen	Display and feedback (see paragraph 2.2)
E. Accessory mode selection	To operate the accessories. The display indicates the currently selected accessory (see paragraph 2.3)
F. Function button	The operation of the function keys is dependent on the value of the function button (see paragraph 2.4)
G. Charge connector	Input for the battery charger
H. Charge plug for the battery charger	Connector for the battery charger
I. Lights button	>2 seconds, lights will be active
J. Hazard lights	>2 seconds, hazard lights will be active
K. 1 Direction indicator left	Switches the left direction indicator on or off
K. 2 Direction indicator right	Switches the right direction indicator on or off
L. "Profile" button	Select driving profile

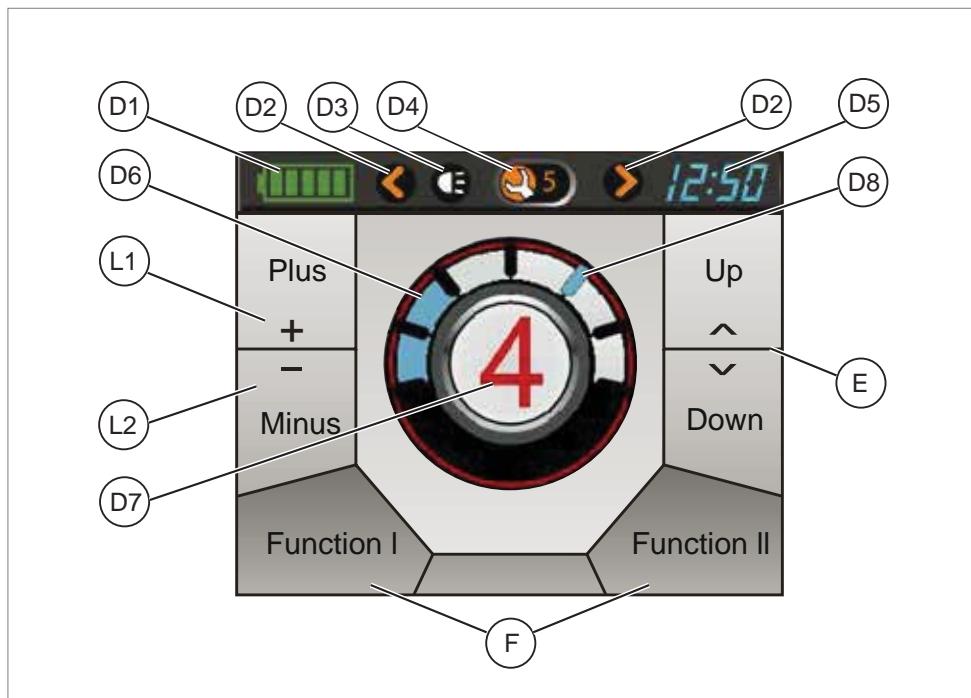
Figure 3

2.2 Display

Part	Function
D1. Battery indicator	Displays the power level of the battery
D2. Active direction indicators	Displays if the direction indicators/hazard lights are on
D3. Active lights	Displays if the lights are on
D4. Fault indicator	Displays a code number if a fault occurs
D5. Clock	Displays the time in 24-hours format
D6. Speed indicator	Displays a graph of the actual speed
D7. Profile	Displays the selected profile
D8. Maximum speed	Displays the maximum speed
E. Accessories	Displays the selectable accessories (see paragraph 2.3)
F. Functions	Displays the selectable functions. Depending on the value of the function button (see paragraph 2.4)
L1. Next profile	To select next drive profile
L2. Previous profile	To select previous drive profile

2.3 Accessories

With the up/down buttons or joystick left/right you can scroll through the different accessories. Accessories are available when they are programmed in the system.

Mode	Select	Active
IRIS Mouse		
ECU 2 (Environmental control unit)		
ECU 1 Mouse		
Lights		
Adjustment options		
Program Screen Settings		

Push the 'accessory mode selection' or 'profile button' to exit the menu item and return to the main menu.

2.4 Functions

Use the function buttons or joystick left/right to select the next or the previous menu item. To control the desired menu item, deflect the joystick front/back.

Function	I		II	
Speed Pot		Slower		Faster
Full Lighting		Left indicator > 2 seconds: lights		Right indicator >2 seconds: Hazard lights
Warning + Hazard		Lights		Hazard lights
ECU Alarm + Toggle ECU Al. (Environmental control unit)		ECU Alarm		ECU Alarm Latched
Warning + ECU alarm		Lights		ECU Alarm

Push the 'accessory mode selection' or 'profile button' to exit the menu item and return to the main menu.

3 Driving the wheelchair with the controller

3.1 Switching the controller on or off

To be able to drive or operate the electric adjustment options of the wheelchair, the controller must be switched on. Press the on/off button (B in figure 2).

Attendant mode is selected when the attendant switch on the attendant control is switched to local. When the attendant mode is selected, the attendant icon will be shown on the screen. During attendant mode all menu navigation is identical to standard mode.

3.2 Driving the wheelchair

Driving an electric wheelchair is done by operating a joystick. Move the joystick forwards and the wheelchair will also move forwards. Steer left and right and the wheelchair will turn.

3.3 Speed

The maximum speed can be controlled by the function button on the controller (F in figure 2). The maximum speed is displayed by the LED lights (D8 figure 3). Speed can be controlled with the joystick while driving. If the joystick is moved a little, the wheelchair will move more slowly.

3.4 Driving Profile

This controller can also be set to make the wheelchair suitable for different driving profiles or environments. For example, selecting the profile for indoors will ensure that the wheelchair will react more 'calmly'. Once outside, the profile can be adjusted to a more 'robust' setting. The currently selected drive profile is shown on the drive profile LED (D7 in figure 3). DX2 Rem 550 allows you to adjust the maximum speed within different profiles or environments.

3.5 Selecting a profile

To switch to a different profile, just press the 'profile' button (L in figure 2). Press the 'profile' button until the most appropriate profile has been selected.

4 Operating the electrical adjustment options

Not every wheelchair has been equipped with electric adjustment options. We make a distinction between four different adjustment options to the seating system:

- Tilt adjustments
- High/low adjustments
- Backrest adjustments
- Legrest adjustments

4.1 Selecting the desired adjustment option

1. Press the 'accessory mode' button (E in figure 2) to select the 'accessory' mode.
2. The display indicates the available accessories (E in figure 3). Use the 'accessory mode' button or joystick to switch between the accessories ('Λ' / right for the next accessory and 'V' / left for the previous accessory).
3. The active accessory will be shown on the display (D7 in figure 3). To control the adjustments select the actuator mode (figure 4) and use the function button (F in figure 2) to selected the desired adjustment option.
4. Moving the joystick forwards and/or backwards will activate the selected electric adjustment option (see figure 5). Move the joystick forwards or backwards until the desired adjustment option has been attained.

Figure 4

	Move joystick backwards	Move joystick forwards
Tilt adjustment	The entire chair will tilt backwards	The entire chair will tilt forwards
High/low adjustments	The entire chair will be raised	The entire chair will be lowered
Backrest adjustments	The backrest will tilt backwards	The backrest will tilt forwards
Legrest adjustments	The legrest angle will increase, the footplate will be raised	The legrest angle will decrease, the footplate will be lowered

Figure 5

5. To return to the 'drive' mode: Press the 'profile' button (L in figure 2).

Note!

If you adjust the seat height by using the high/low option or use the electrical tilt adjustment 0 - 45°, the speed will be reduced due to safety reasons.

5 Lights

With the DX2 Rem 550 it is possible to control the following lights:

- Lights (I in figure 2)
- Hazard Lights (J in figure 2)
- Direction indicators (K1 & K2 in figure 2)

You can also push the accessory mode button and select the light option. To control the lights use the joystick (figure 6):

- forwards: lights
- backwards: hazard lights
- left: direction indicator left
- right: direction indicator right

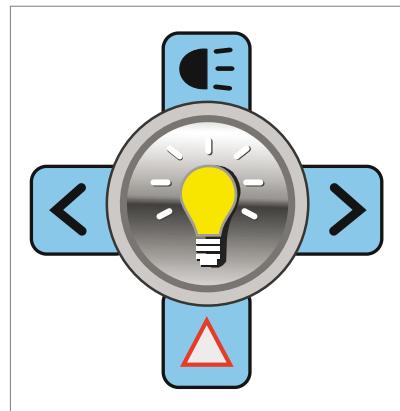
Figure 6

6. Troubleshooting

If the wheelchair will not function while the batteries are fully charged, check the following points before consulting your dealer:

- Switch the controller off and then switch it on again. Check to see if the malfunction has been solved.
- Check if the free wheel switch was switched to Drive.
- Check if the joystick was in the 0 position when the controller was switched on. In other words, the joystick must not be moved when the controller is being switched on or off.

Malfunctions list

An extended malfunction list can be found in the appendix of the service manual (for qualified specialists only). The service manual can be found on www.SunriseMedical.eu

7. Locking the controller

To lock the controller:

- Press the on/off button for 4 seconds when the DX system is turned on.
- The DX system turns off in a locked state.

To unlock the controller:

- Press the on/off button. The joystick will show a lock on the display.
- Press the horn button twice within 10 seconds. The DX System will turn on normally.

8 Technical specifications

Operating voltage range:	18V - 32V DC (nom. 24V)
Charger rating:	max 12A RMS Continuous limited by DXBUS Rating
Quiescent Current:	< 0.25mA Off, typically 200mA On
Tube mount diameter	Min 15mm, Nom 19mm, Max 22mm
Force required to operate joystick	Less than 2.2N
Operating forces	Less than 8N
Protection rating:	IPx4
Operating Temperature Range:	-25°C to +50°C
Storage Temperature Range:	-40°C to +70°C
Operating Humidity Range:	0 to 95% RH

EMC tested on sample wheelchair:

	ANSI/RESNA WC/Vol:2 - 1998 Sec 21 EN12184: 1999 Sec 9.8.1-3
--	--

9. Technical diagrams

9.1 Technical diagram

The technical diagrams can also be found on the cover of the specific electronic component.

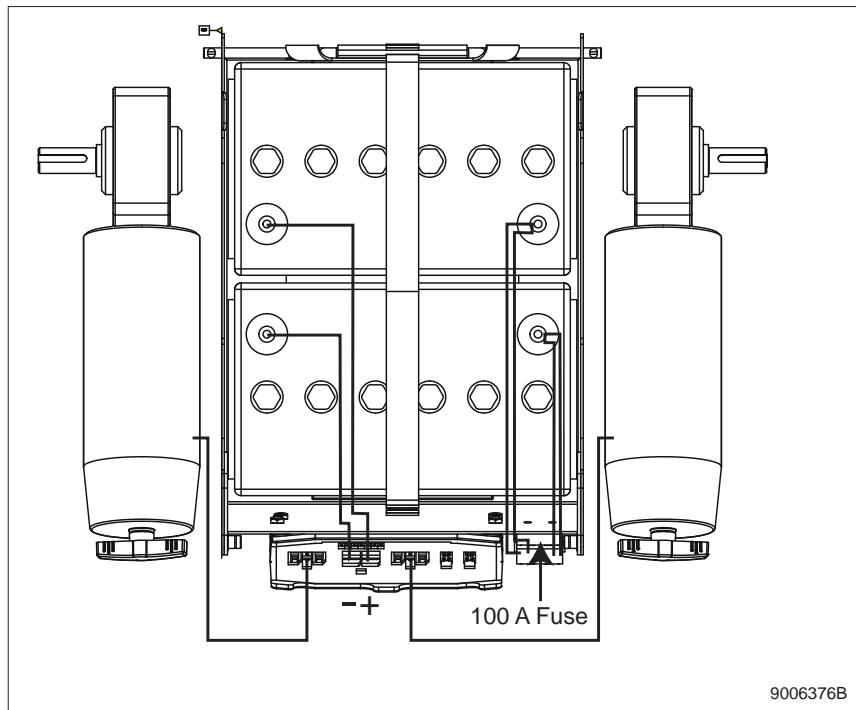
Figure 7

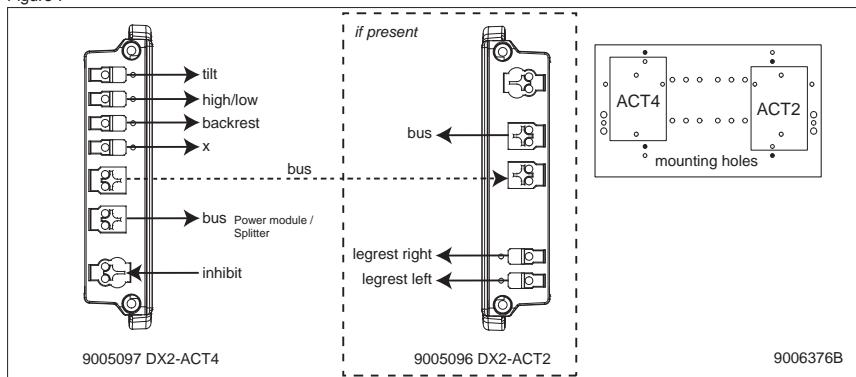
Figure 8

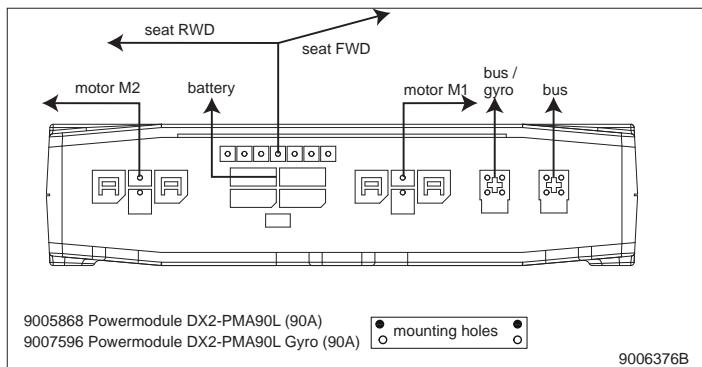
Figure 9

9.2 Technical diagram battery charger

The controller's standard configuration includes a '3-pin connection'. Ensure that the battery charger is properly connected so that the 'negative pole' and the 'inhibit' are connected, enabling the system to prevent the wheelchair from moving when the battery is being charged.

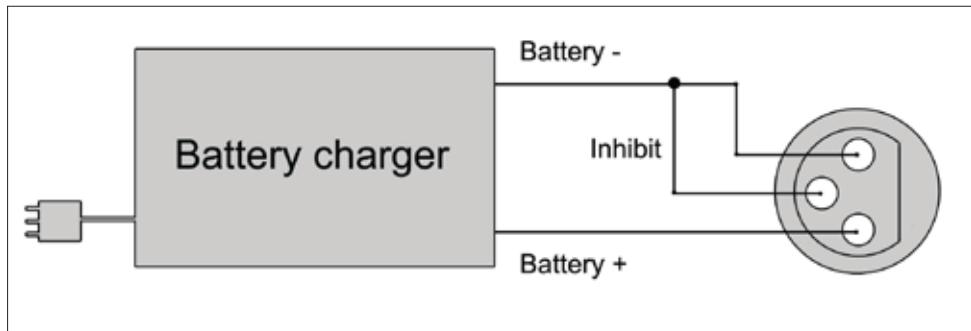
Figure 10

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Sunrise Medical AB