JAY[®]

Jay[®] Fit Cushion

User Instruction Manual & Warranty

Cuscino *Jay*[®] Fit

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⚠ WARNING

Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backward, potentially resulting in injury. Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

I. INSTALLATION INSTRUCTIONS

The Jay Fit Cushion is for wheelchair users. The seat is easy to adjust based on a child's changing needs. When installing or adjusting components, tighten all fasteners to 2.3 N-m (20 in.-lbs.).

A. INSTALLING THE JAY FIT CUSHION HOOK RETAINERS

1. Remove any seat upholstery from the wheelchair seat rails.
2. Install one Hook Retainer part #284 toward the rear of each seat rail (Figure 1). The bolt holes used to secure the Hook Retainers should be below the seat rail. The extension on the top of the Hook Retainers should extend forward.
3. Place a nut into the hexagonal depression on the Hook Retainers, toward the inside of the wheelchair seat frame.
4. From the outside of the wheelchair seat frame, using the appropriate length bolt for the installation, insert the bolt through the holes in the Hook Retainers into the nut and tighten with a Phillips screwdriver.

NOTE– The Jay Fit Cushion hooks supporting the seat should slide under the extension on the top of the Hook Retainers.

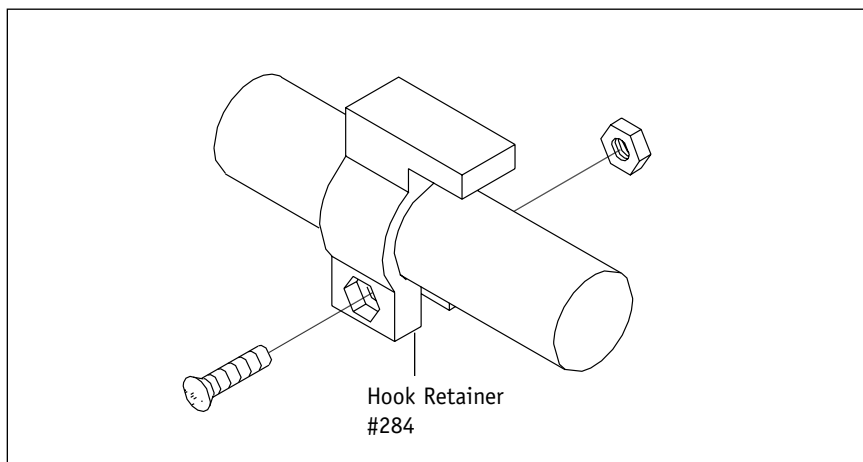
Figure 1

B. INSTALLING THE HOOK STOPPERS

1. Install the Hook Stopper part #283 toward the front of one seat rail (Figure 2). The bolt holes used to secure the Hook Stopper should be below the seat rail.
2. Place a nut into the hexagonal depression on the Hook Stopper toward the inside of the wheelchair seat frame.
3. From the outside of the wheelchair seat frame, using the appropriate length bolt for the installation, insert the bolt through the holes in the Hook Stopper into the nut and tighten with a Phillips screwdriver.

NOTE– One of the hooks supporting the front of the seat should rest behind the Hook Stopper. There should be a space between a front Seat Hook and the Hook Stopper.

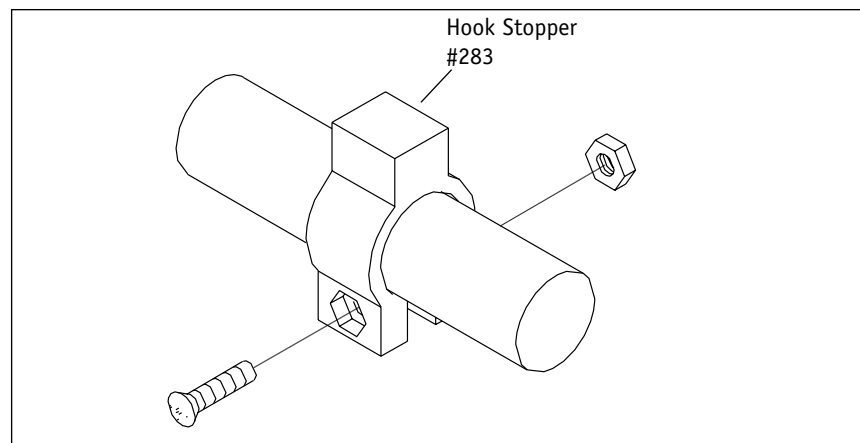
Figure 2

C. INSTALLING THE ATTACHING HARDWARE ASSEMBLIES

1. Identify the attaching hardware assembly.
2. Turn the Jay Fit Cushion over exposing the underside and the tracks. The attaching hardware assemblies may be attached at any position on the track.
3. Slide the channel nuts into the track as shown (Figure 3) and hand tighten so that the assemblies may be adjusted to the desired final position. Repeat for each assembly.
4. Place the Jay Fit Cushion onto the wheelchair's seat rails and center it between the frame, adjusting the assemblies on the seat to the desired depth and width. (Each track will have two assemblies.)
5. Carefully remove the Jay Fit Cushion and tighten all nuts securely.

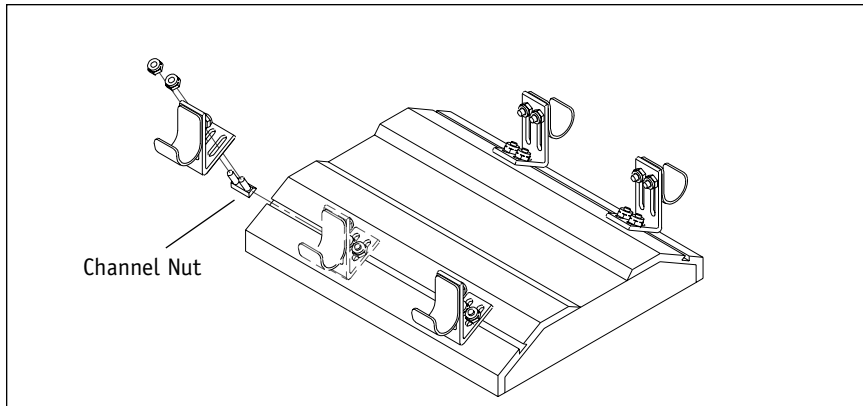
Figure 3

D. ADJUSTING DROP HOOK HEIGHT

The Drop Hooks may be adjusted up or down 5.1 cm (2").

To adjust the brackets for height:

1. Loosen the two nuts attaching the Drop Hook to the Mounting Bracket (Figure 4).
2. Slide the Drop Hook to the desired position on the Mounting Bracket and re-tighten the nuts.
3. Repeat for all Drop Hooks.

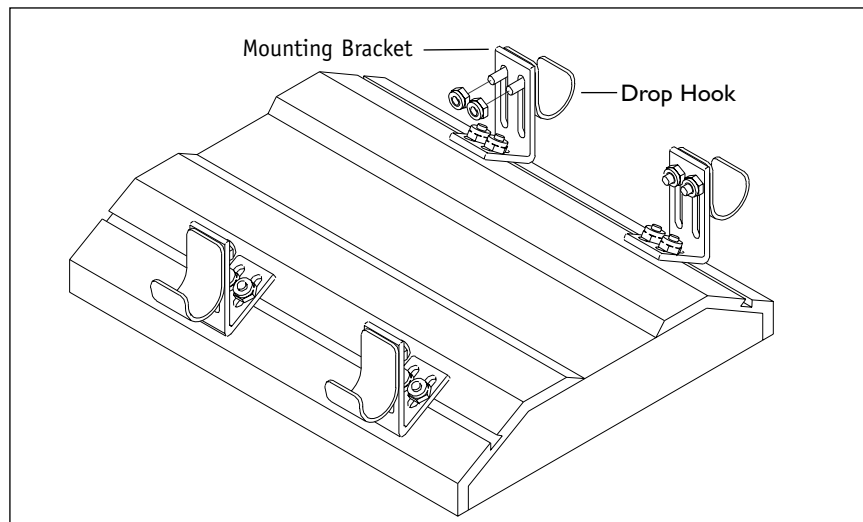
Figure 4

II. INSTALLING & REMOVING THE CUSHION

1. To install the seat (Figure 5), place the rear hooks on the seat rails and slide the seat back while lowering the front.

NOTE– The rear hooks should slide under the Hook Retainers and one of the front hooks should rest behind the Hook Stopper.

2. To remove the seat, lift the front of the seat and pull forward and up.

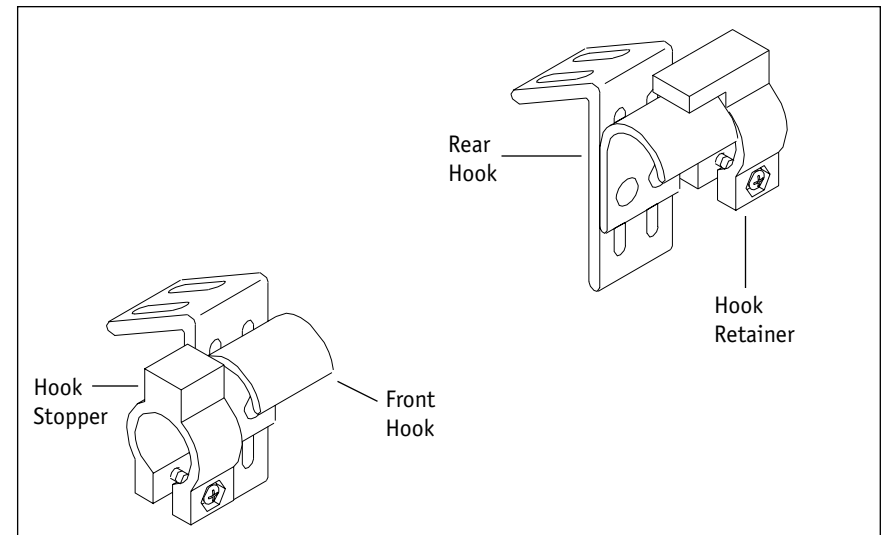
Figure 5

III. FITTING THE SEAT WELL COMPONENTS TO THE CLIENT

A. FIT CUSHION WITH FOAM OPTION

1. Remove the Jay Fit Seat outer cover.
2. Unzip the inner cover located at the back of the seat and fold it back so that you may access the inner seat base well area.
3. Insert the desired number of anterior well reducers into the seat well (Figure 6).

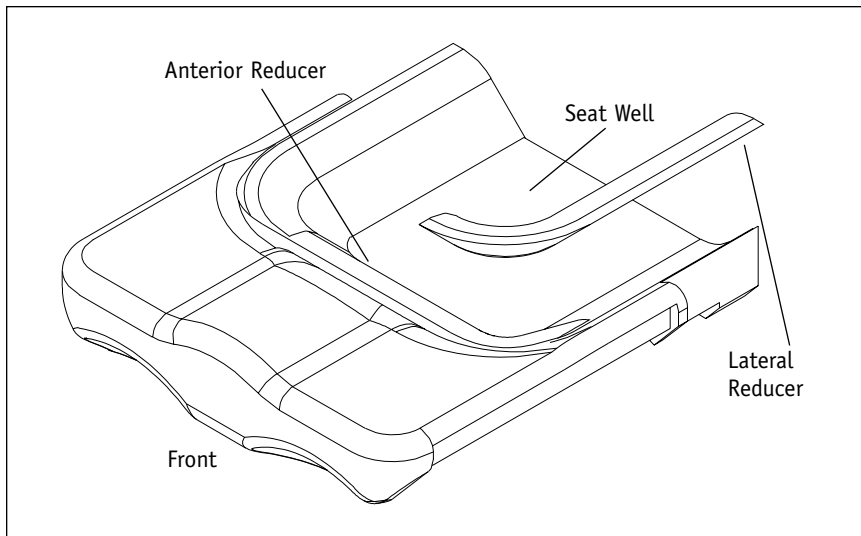
Figure 6

NOTE– The anterior well reducer helps to prevent the pelvis from tilting into a posterior position. Each reducer is 2.54 cm (1") thick, and attaches to a Velcro track. The seat base is notched 7.6 cm (3") in the rear so that it may be positioned behind the back posts. As the child grows in femur length, the seat may be moved forward and anterior well reducers may be added to maintain pelvic positioning.

4. Add lateral well reducers to the well if necessary (typically one left and one right) for lateral stability.

NOTE– The lateral well reducers are 2.54 cm (1") width. Both the lateral and anterior well reducers may be modified as needed.

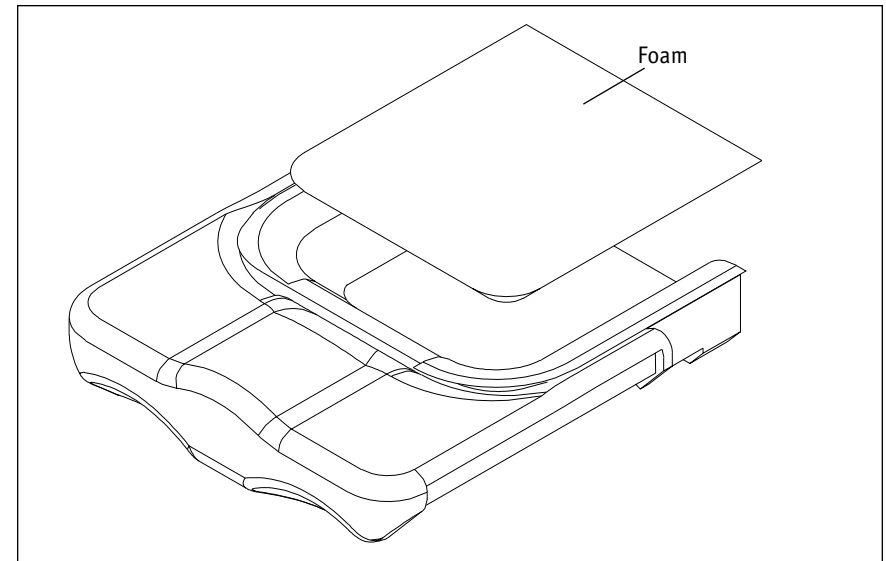
Figure 7

5. Insert layers of foam into the seat well (Figure 7).

NOTE– Two to three layers of foam are recommended, with softer layers on the top to help the child lower into the well for best clinical benefit.

6. Once the number of desired well reducers are in place, cut the foam to match the well size.
7. Close the zipper.
8. Replace the outer cover.

CONTOURED PELVIC AND THIGH SUPPORT OPTION

The contoured pelvic and thigh supports add lateral stability. Available in 5.1 cm (2") or 7.6 cm (3") heights, these supports are adjustable for depth, width, and angle (Figure 8).

NOTE– The length of these supports will be determined by seat size.

To adjust these supports:

1. Remove the outer cover.
2. Adjust the width or angle of the pelvic and thigh supports by loosening the top nuts (Figure 9) for each support, positioning and re-tightening the nuts.
3. Replace the outer cover.

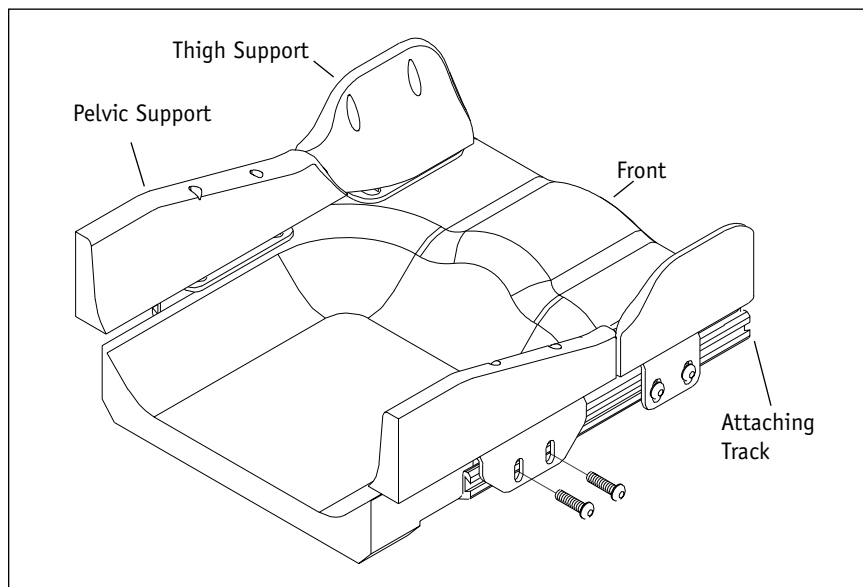
Figure 8

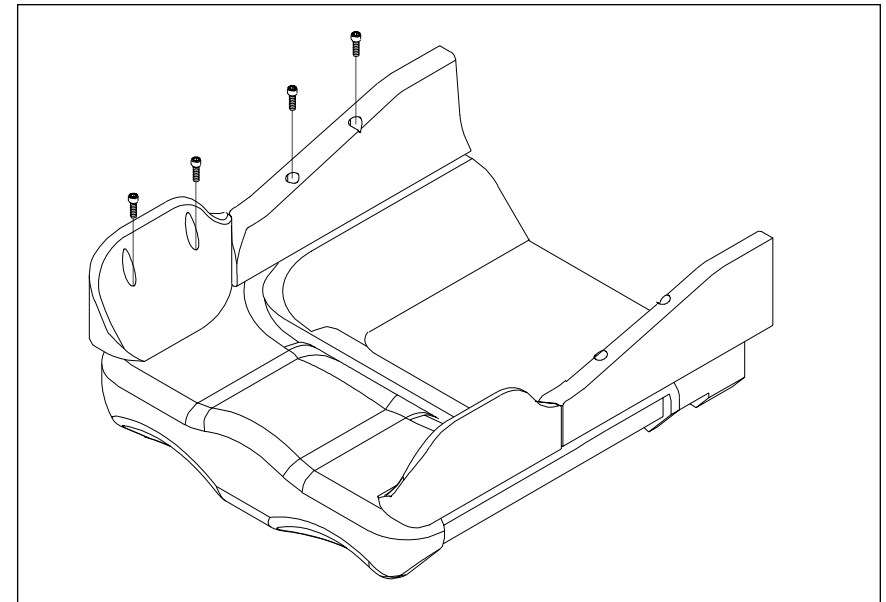
Figure 9

NOTE– Each order for lateral and pelvic supports includes a width expansion kit

(#468111). This kit (Figure 10) includes:

- (2) Spacer blocks (#463241)
- (4) 1/4 X 28, .81 screws (#XM2148)
- (2) Bracket extensions (#XM2030)

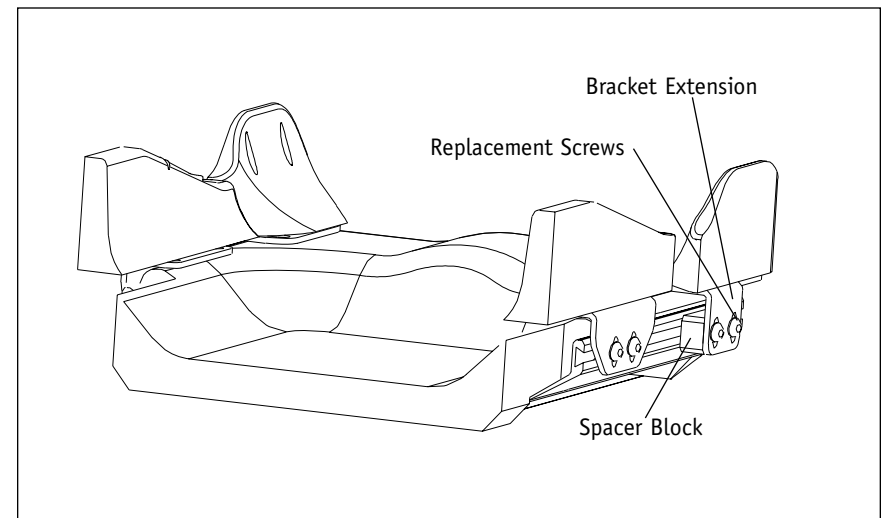
Figure 10

EASY MAINTENANCE

Keeping your Jay Fit Cushion in good condition may ensure many years of use. The foam in the seat well should require no regular maintenance. Monthly cleaning is recommended. To clean the cushion remove the outer cover and wipe the inner cover and lateral pelvic and thigh supports with warm water and soap. The outer cover may be machine washed and dried. Wash in warm water. Do not bleach. Do not dry clean or steam autoclave the outer cover.

With the outer cover removed, inspect the lateral pelvic and thigh supports to see that they have not loosened from the track. If the supports have loosened, re-tighten with a 5/32" hex wrench or see your authorized supplier.

Re-cover the cushion with the label on the side and the content labels on the back.

24 MONTH LIMITED WARRANTY

This product has been carefully inspected and tested to provide peak performance. This product is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. **This warranty does not apply to punctures, tears or burns to the cushion's removable cover.**

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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©2001, Sunrise Medical 12.01
Sunrise Medical Ltd.
Sunrise Medical Pty. Limited