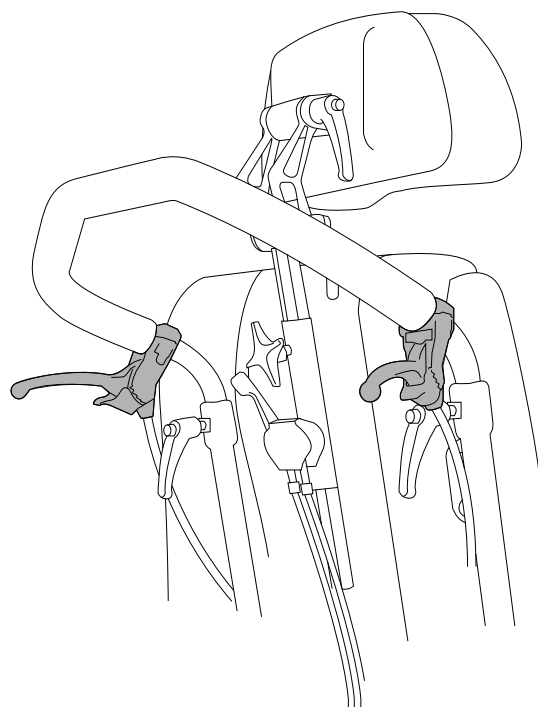
ASSEMBLY INSTRUCTION


Drum brake

Ledsagerbrems

Description / Beskrivelse	Item no./ Art. nr.	
20" Pneumatic / Luft	☐	84416
22" Pneumatic / Luft	☐	84436
24" Pneumatic / Luft	☐	84456
20" Compact / Kompakt	☐	84419
22" Compact / Kompakt	☐	84439
24" Compact / Kompakt	☐	84458
Parts for dum brake / Deler til ledsagerbrems - Emineo Basic	☐	3409994

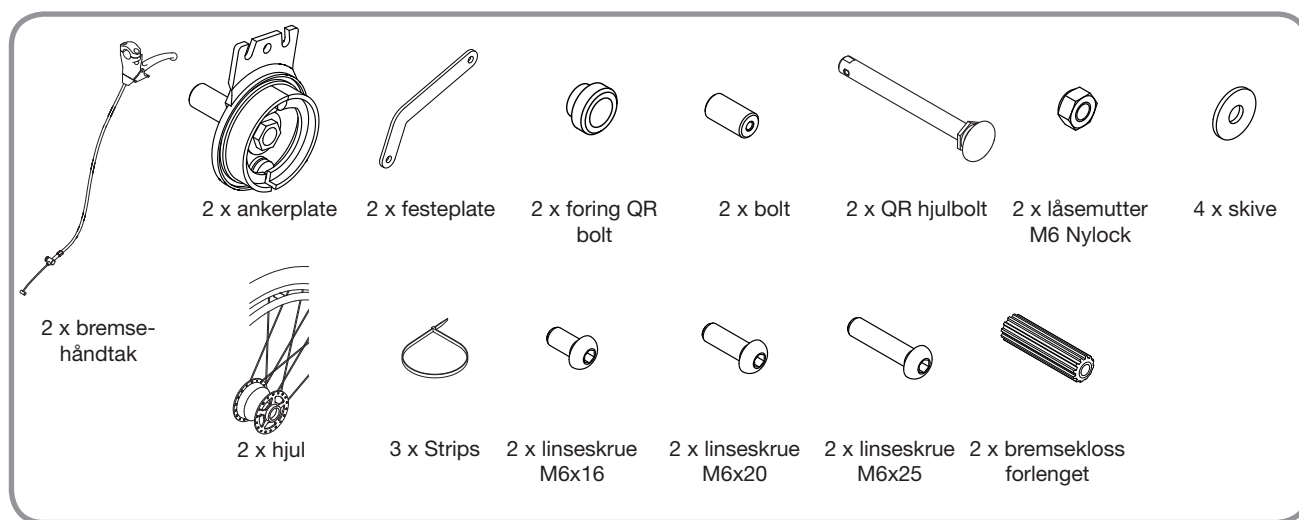EN

Preparatory work

- 1 Put on the push handles.
- 2 Remove the wheel. Remove the existing casing. In order to do this, use a 27 mm wrench to work loose the nut with washer. Use a 16 mm wrench to unscrew the casing.

Keep the washer ^(A) for use under paragraph 4.

- 3 **Assembling the anchor plate**
In order to assemble the anchor plate in the foremost hole pattern of the wheel base, see figure 3a. See figure 3b for assembling in the rear hole pattern.
- 4 **Fixing the anchor plate to the wheel base**
In order to assemble the anchor plate in the foremost hole pattern of the wheel base, see figure 4a. Reuse the washer ^(A) from paragraph 2. See figure 4b for assembling in the rear hole pattern.

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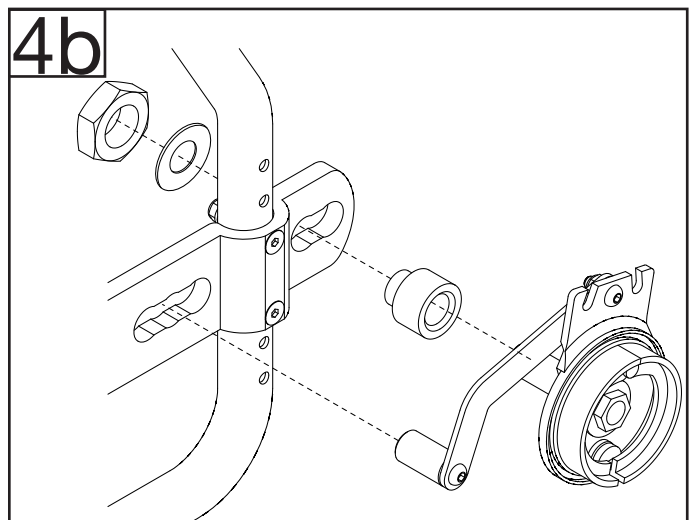
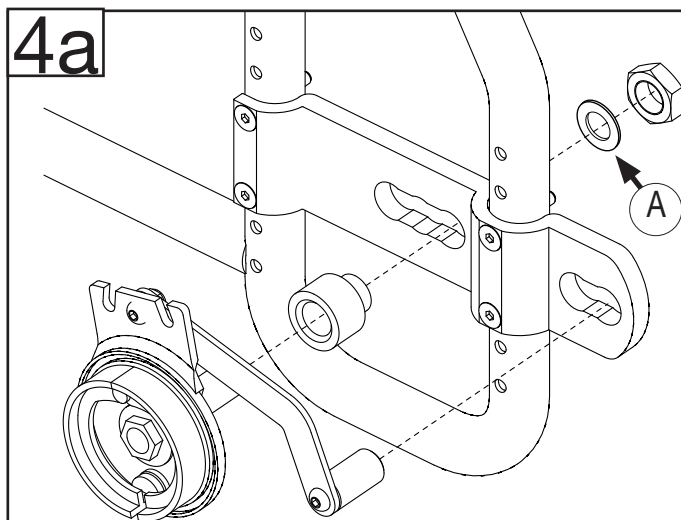
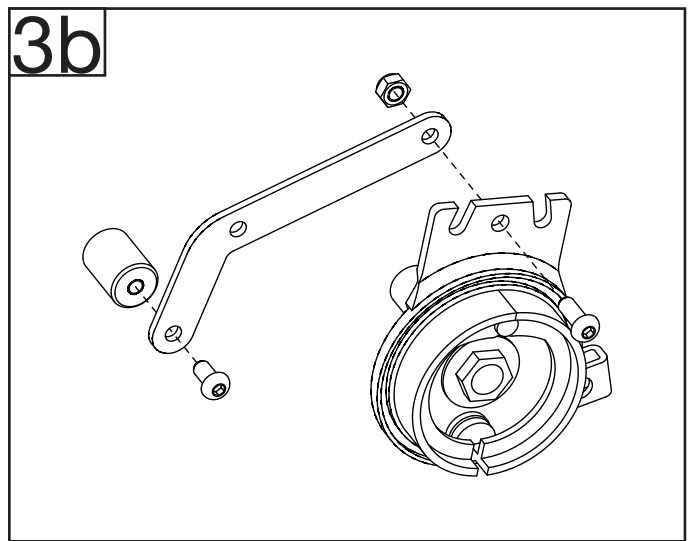
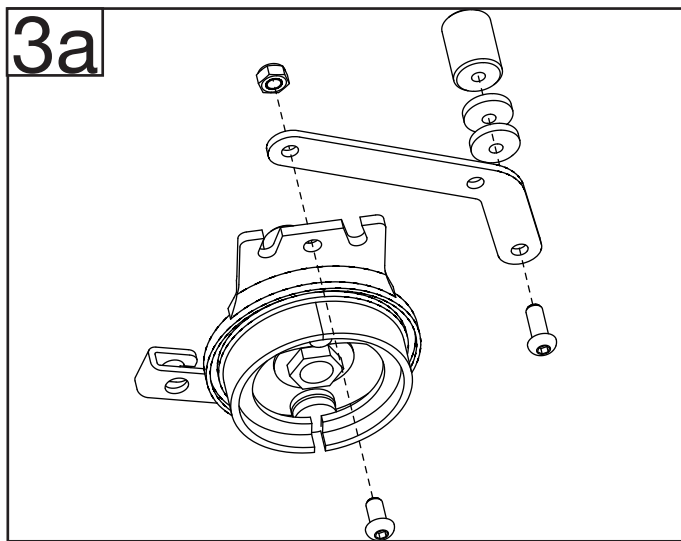
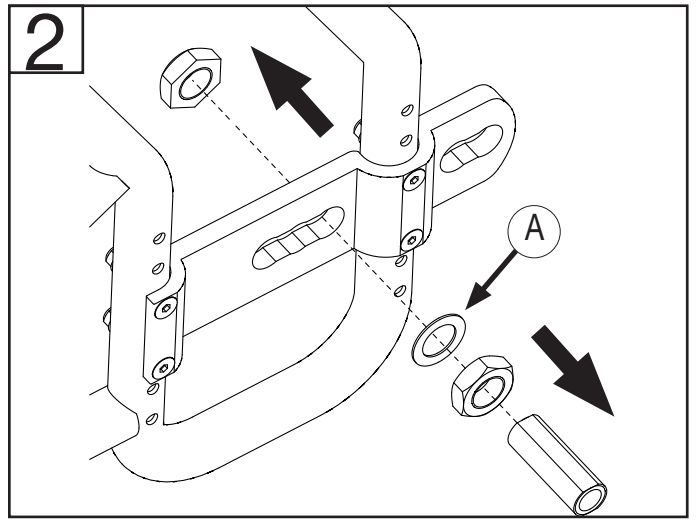
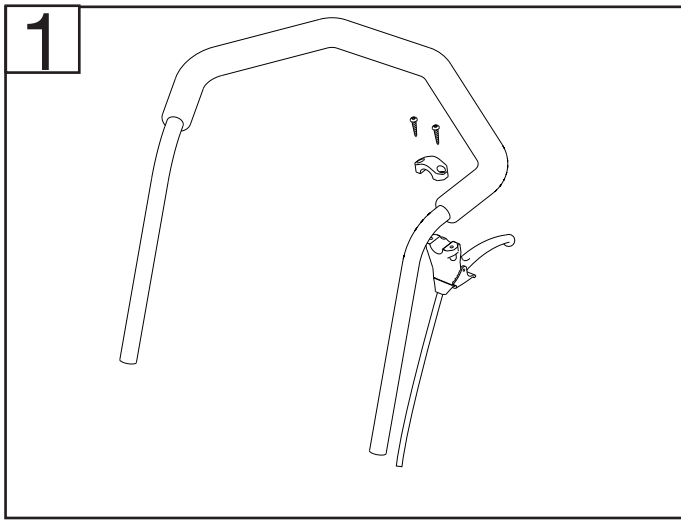
- 5 Use a 17 mm socket wrench to tighten the casing.
- 6 Fix the wire  to the brake handle.
- 7 Screw the wire stretcher  to the most slack position. Fit the wire stretcher to the slot in the anchor plate.

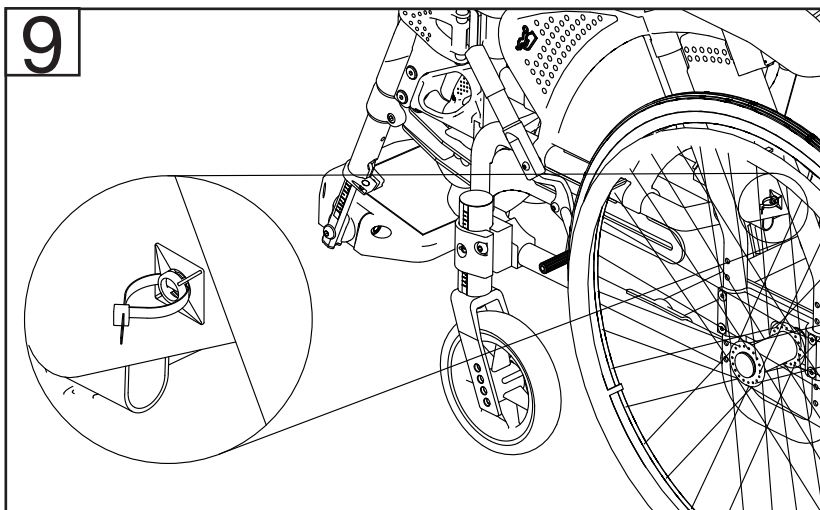
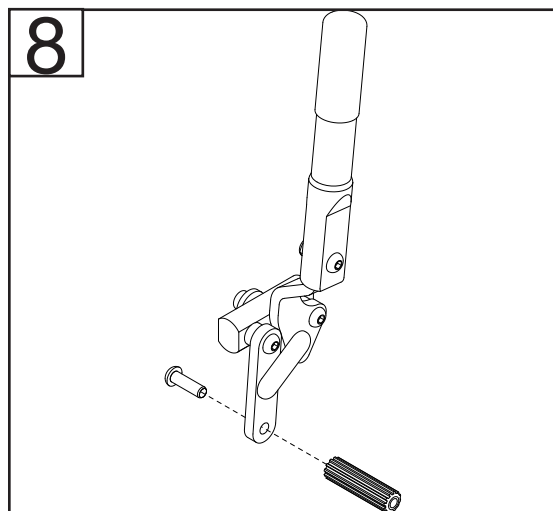
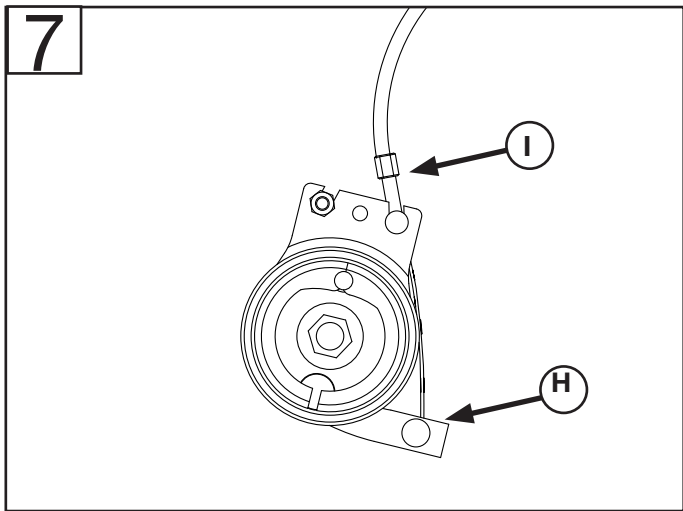
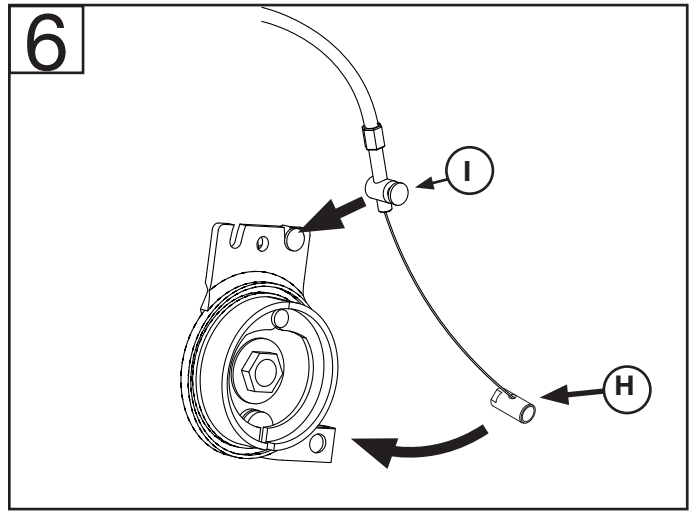
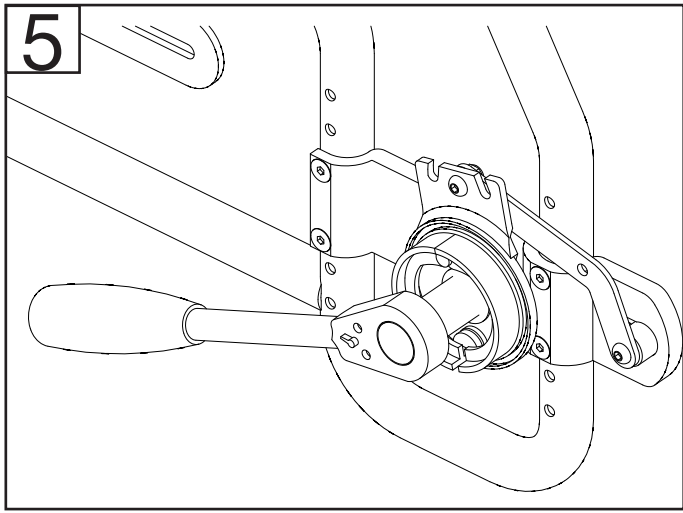
To adjust the drum brake, screw out the wire stretcher step by step. Mount the wheel and test the brake by spinning the wheel and squeezing the brake handle. The wheel should spin easily.

- 8 Unscrew the brake block as displayed in figure 9. Replace with the extended brake block.
- 9 Use strips to fasten the brake-wire. Glue the wire fixing to the side cover which is located by the armrest. Use trips to assemble the brake-wires behind the back.

- 5 Trekk til drivhjulshylsa med en 17 mm pipenøkkel.
- 6 Fest vaierenden  til bremsehendelen.
- 7 Skru kabelstrammeren  til slakkeste posisjon og fest den til sporet i ankerplata.

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Adjusting the seat height and the centre of gravity

11 Seat height

In order to move the anchor plate up or down; unscrew the four screws attaching the wheel block to the frame. It is not necessary to remove the anchor plate.

12 Centre of gravity

Use a 17 mm socket wrench and a 24 mm spanner to loosen the casing.

- 13 Move the anchor plate forward or backward. In order to use the opposite hole pattern turn the anchor plate. The rear hole pattern represents the best anti-tipping position.

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Sunrise Medical HCM B.V.