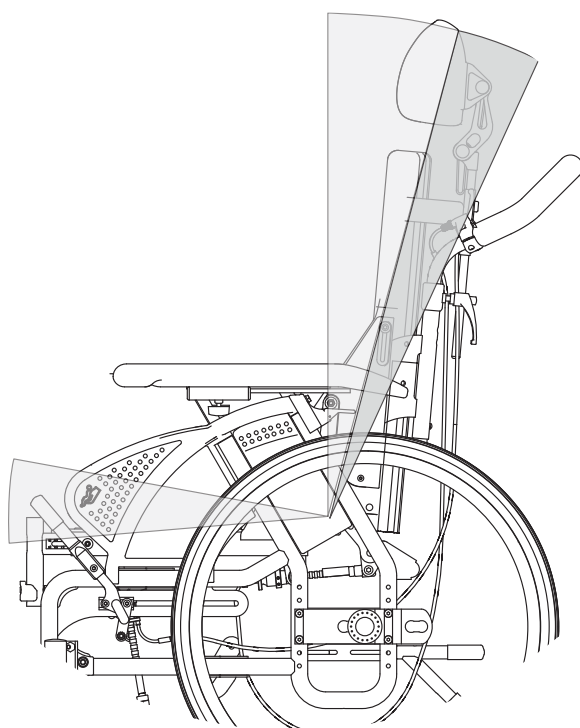
ASSEMBLY INSTRUCTION

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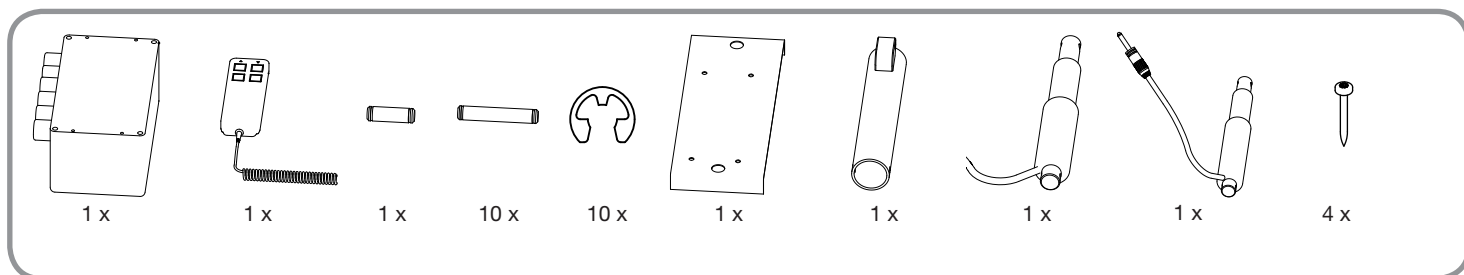
3019505

EN

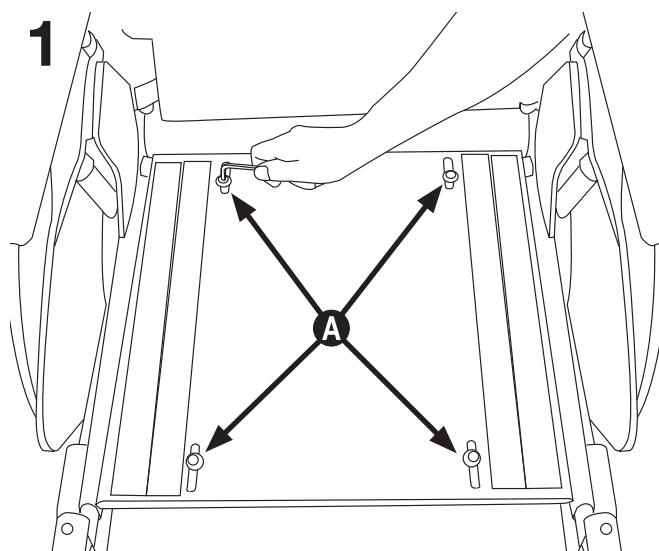
Preparations – disassembling of adjustment mechanism for seat tilt and back

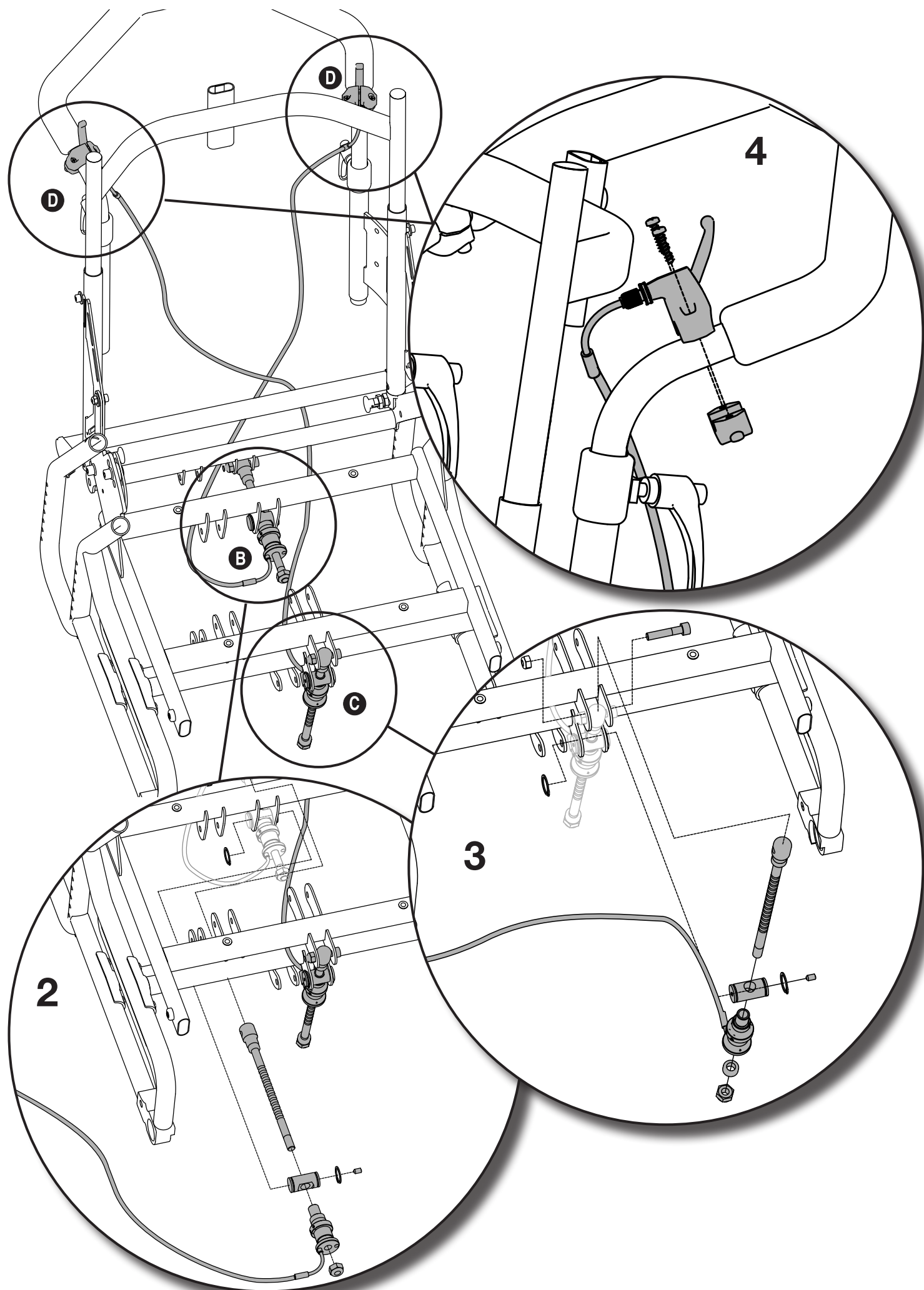
- 1 Disassemble the seat plate by unscrewing the four screws **A** (keep the screws).
- 2 Disassemble the adjustment mechanism for back **B**.
- 3 Disassemble the adjustment mechanism for seat **C**.
- 4 Disassemble wires and handles **D**.

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
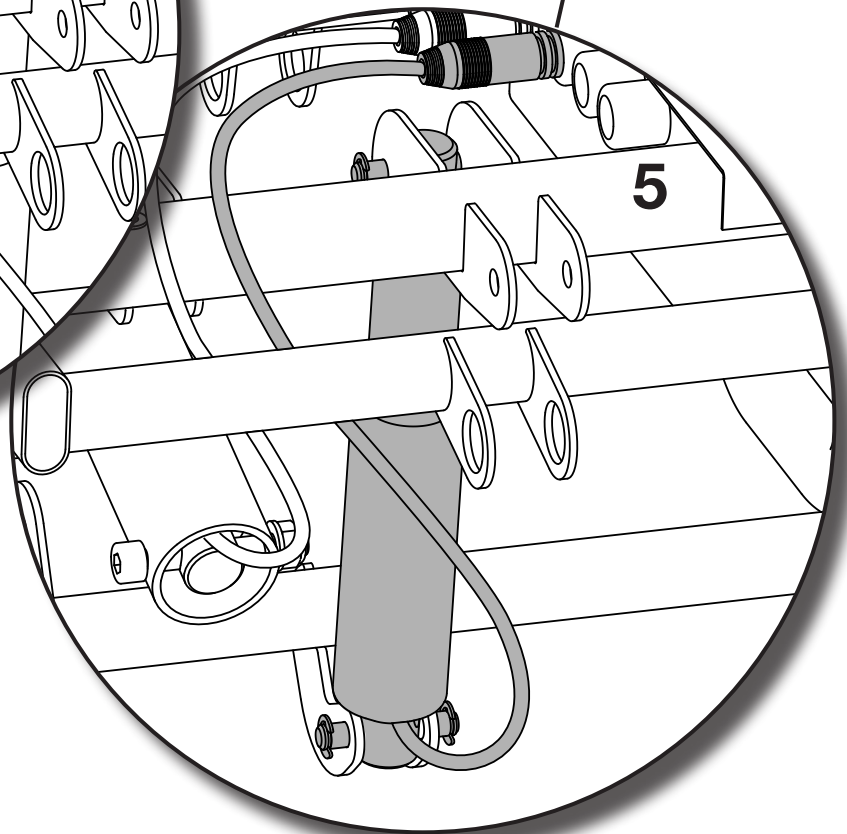
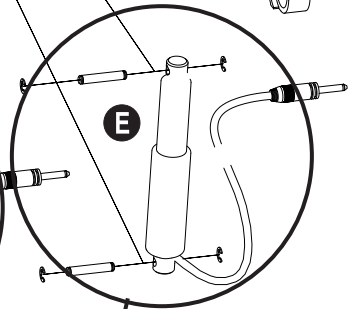
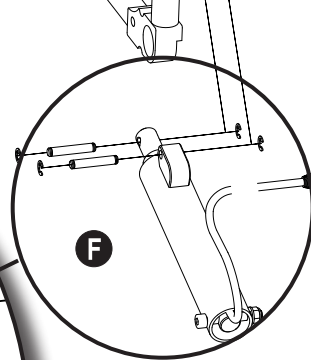
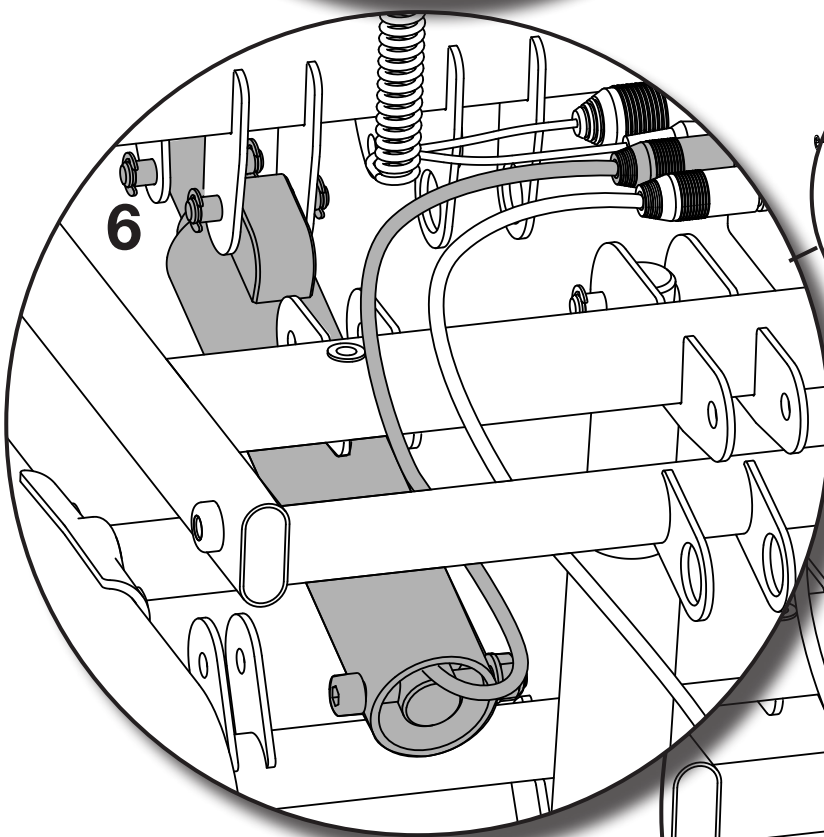
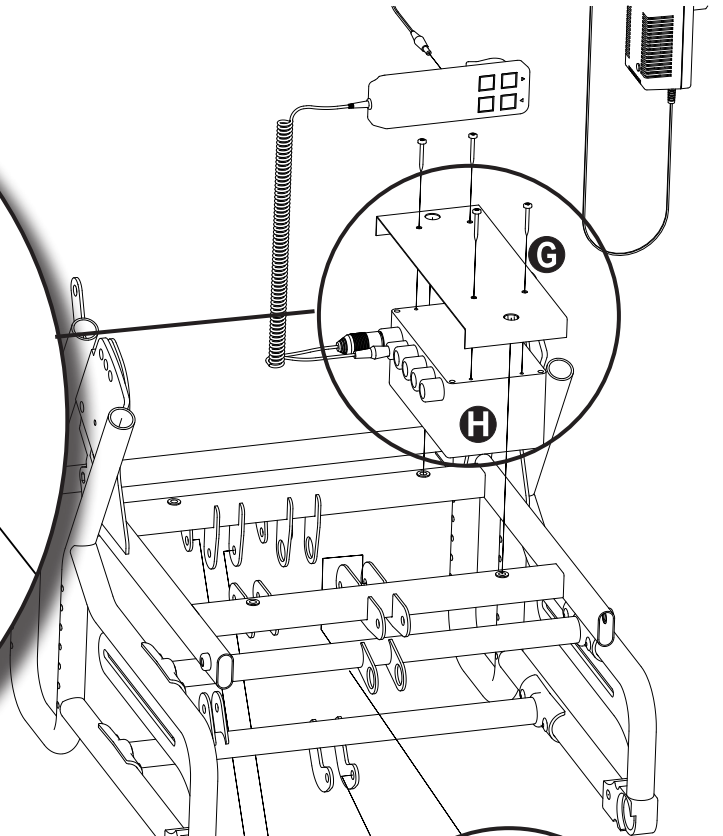
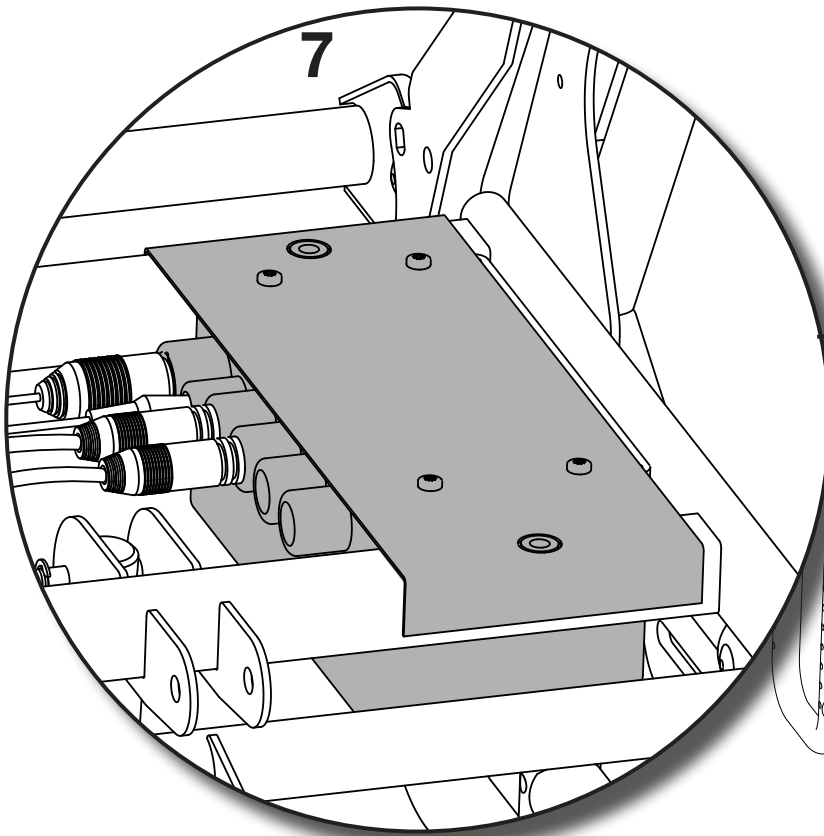
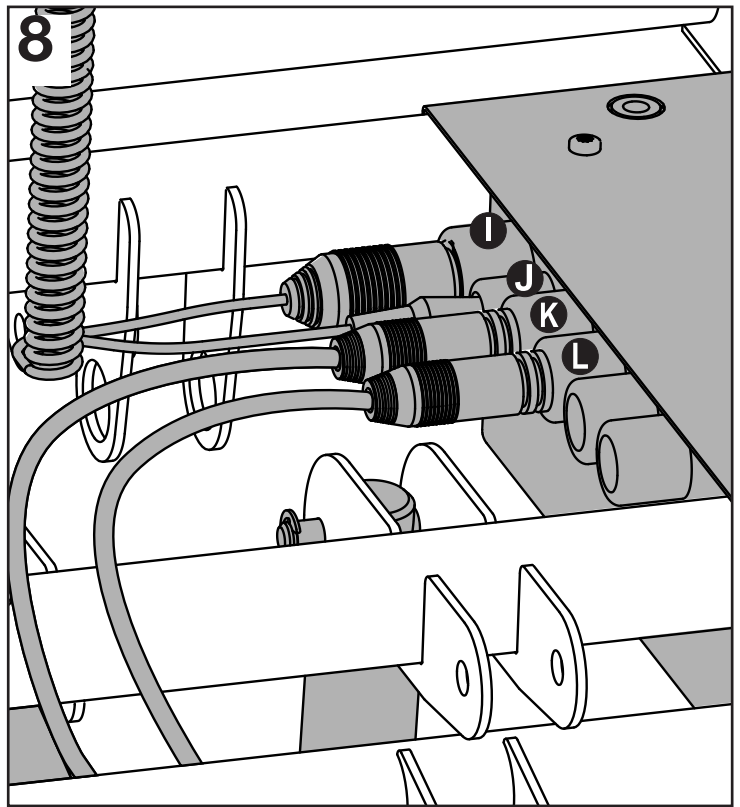
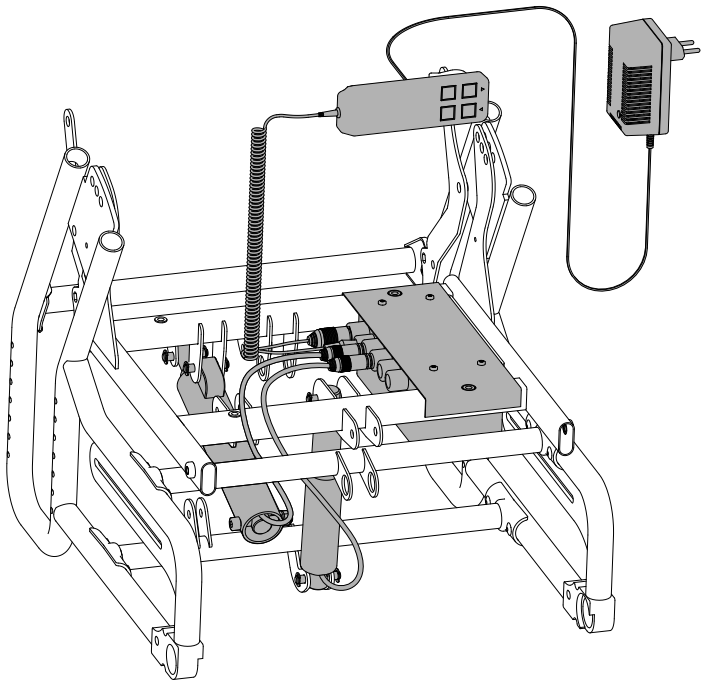
EN

- 5 Assemble seat actuator **E**.
- 6 Assemble back actuator **F**.
- 7 Screw together fixation plate **G** and control unit **H**.
Place the jointed fixation plate and control unit onto the seat frame, see illustration. (The control unit will be fixed to the frame when you reassemble the seat plate).
- 8 Lay the cables to the control unit
 - I** hand control
 - J** cable for charger
 - K** actuator back
 - L** actuator seat tilt
- 9 Remount the seat plate, ensure that the control unit is fixed to the seat frame while tighten the screws.

NO

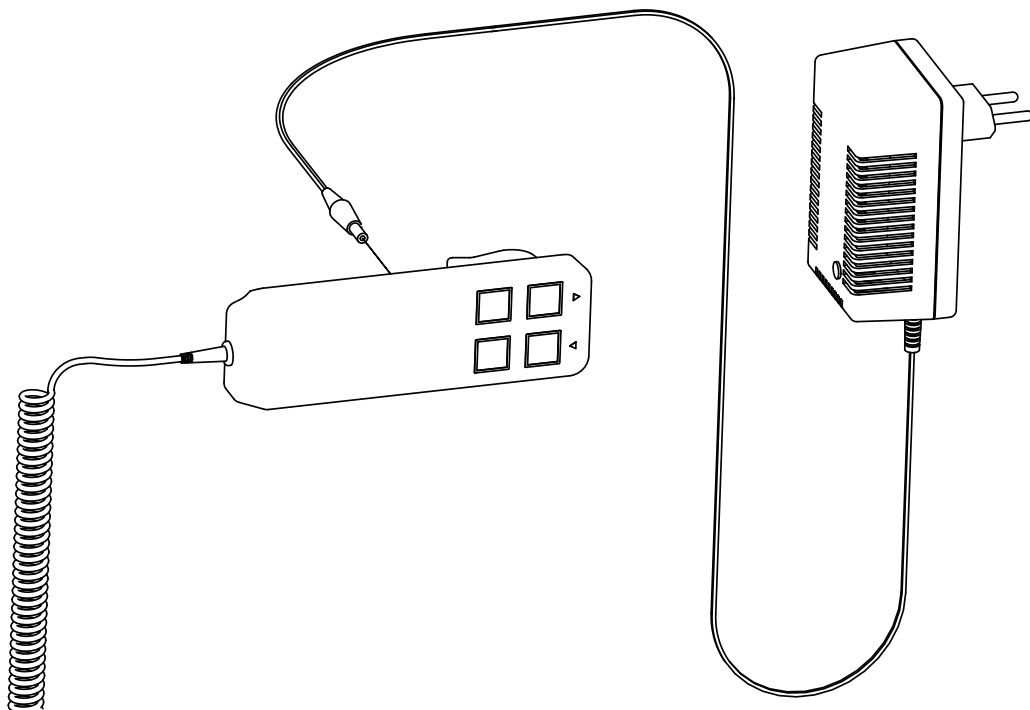- 5 Monter aktuator for sete **E**.
- 6 Monter aktuator for rygg **F**.
- 7 Skru sammen festeplate **G** og kontrollboks **H**.
Plasser festeplate med kontrollboks på seterammen som vist i figuren (Kontrollboksen festes til rammen når du skrur fast seteplata).
- 8 Trekk kabler til kontrollboksen
 - I** håndkontroll
 - J** ladekabel
 - K** aktuator for rygg
 - L** aktuator for setetilt
- 9 Monter seteplata, påse at du samtidig fester kontrollboksen til seteramma.

EN

Charging procedure

Insert the charger plug in the hand control. Connect the charger to the power supply (230 V). The indicator on the charger shines green when the battery is fully charged.

Do not use the wheelchair while charging. Charge the batteries when the wheelchair is not in use.

Washing

If you flush the wheelchair with a hose or a high-pressure jet spray, you must first disassemble the actuators and control box.

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Sunrise Medical HCM B.V.