


ASSEMBLY INSTRUCTION

Wheelbase Hjulblokk

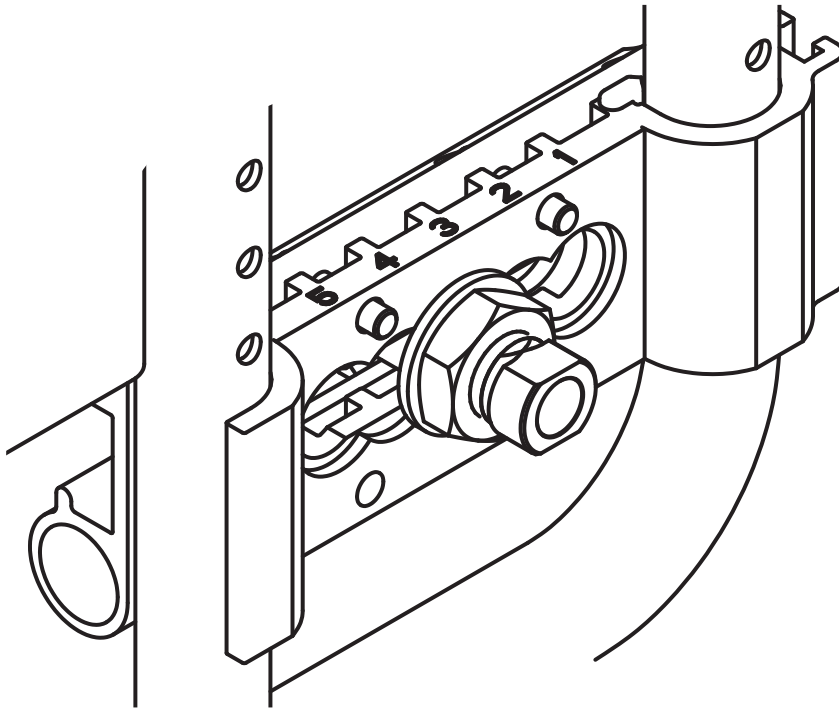



Description

Wheelbase One-tool / Hjulblokk One-tool

Article number

3307159






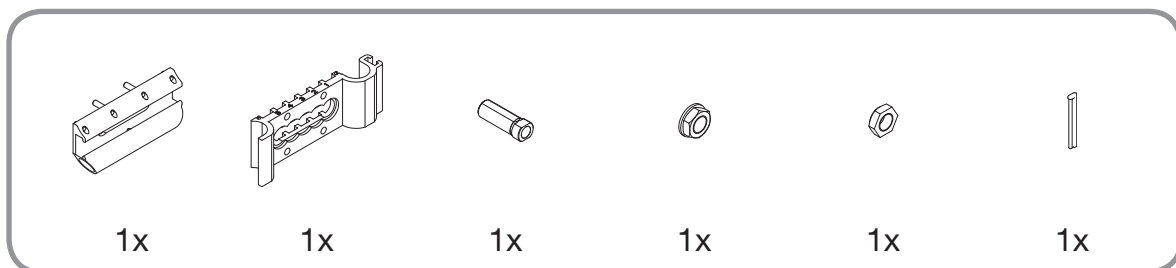
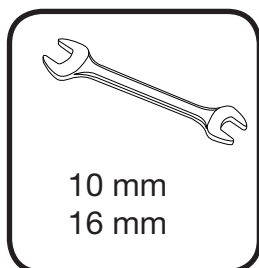


EN

- 1 Place the outer wheelbase onto the frame.
- 2 In order to fix the outer wheelbase to the frame, slide the lockwasher (A) into the notch on the outer wheelbase.
- 3 Squeeze the inner and the outer wheelbase together. It is important to ensure that the pins in the wheelbase go into the holes in the frame. Read the scale on the frame for correct seat height. Slide the lock nut (B) into the notch of the inner wheelbase.
- 4 Screw the QR adapter into the nut with washer. Pull the QR adapter into 1st or 5th track in the wheelbase, and use a 16 mm wrench to fasten. Screw until the end of the QR adapter is only just visible outside the lock nut.
- 5 Slide the QR adapter to correct position according to the centre of gravity. (The parallel surfaces of the QR adapter are to be horizontal for you to be able to slide the QR adapter into the wheelbase). It is important to ensure that the QR adapter lies in the track before you use a 27 mm wrench to tighten the nut with sleeve.

NO

- 1 Plasser ytre hjulblokk på rammen.
- 2 Skyv låsebrikken (A) ned i sporet i ytre hjulblokk for å feste ytre hjulblokk på rammen.
- 3 Klem indre og ytre hjulblokk sammen. Det er viktig at tappene i hjulblokkene går inn i hullene i rammen. Avles riktig setehøyde på skalaen på rammen. Skyv låsemutteren (B) inn i sporet i indre hjulblokk.
- 4 Skru hjulhylsen inn i mutteren med skive. Skyv hjulhylsen inn i spor 1 eller 5 i hjulblokken og skru den til med en 16 mm fastnøkkel. Skru hjulhylsen litt utenfor låsemutteren.
- 5 Skyv hjulhylsen til riktig posisjon i forhold til tyngdepunkt. (De parallelle flatene på hjulhylsen må være horisontale for at du skal kunne skyve hjulhylsen i hjulblokken). Påse at hylsen legger seg ned i sporet før du benytter en 27 mm fastnøkkel og skrur til mutteren med skive.

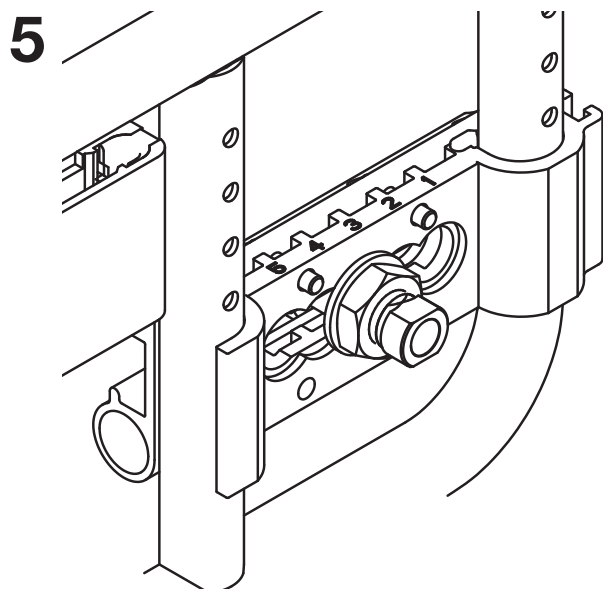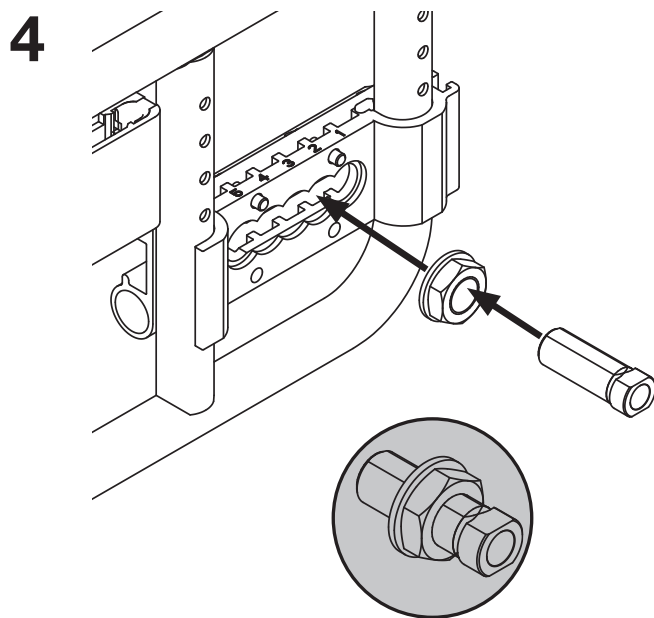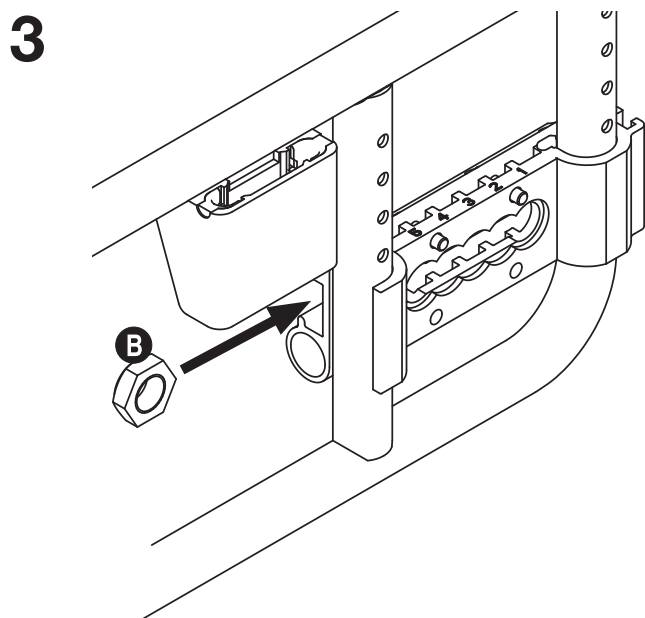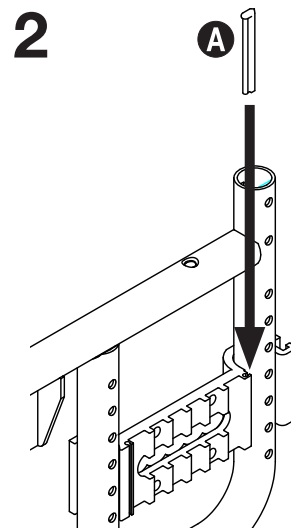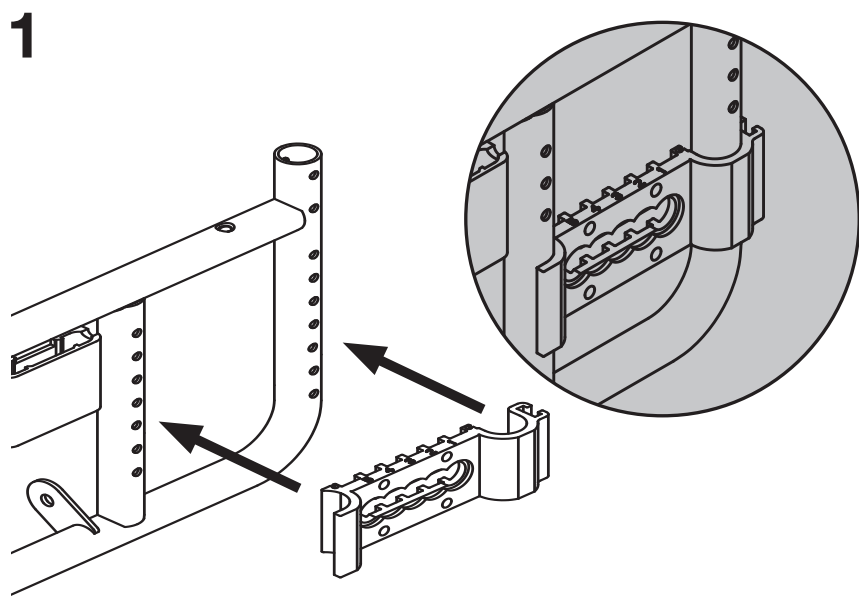



Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands

T: +31 (0)492 593 888
customerservice@sunrisemedical.nl
www.SunriseMedical.eu