



ASSEMBLY INSTRUCTION

Fastening bracket for castor housing Feste for lagerhus

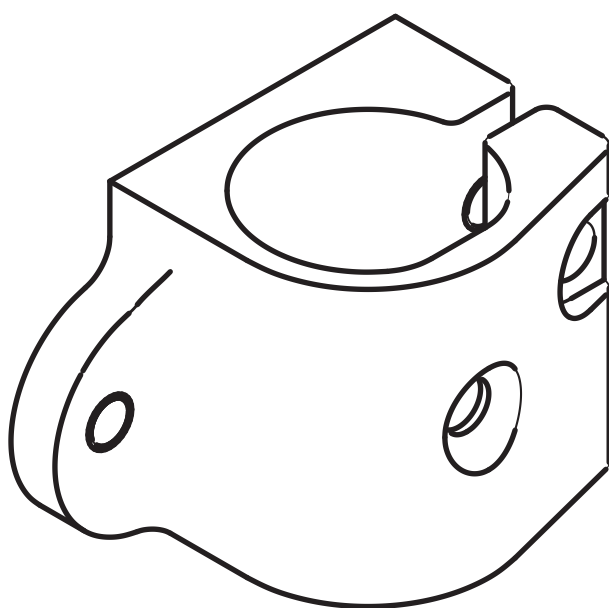


Description	Article number	
RHS / HØ	☐	3307030
LHS / VE	☐	3307031



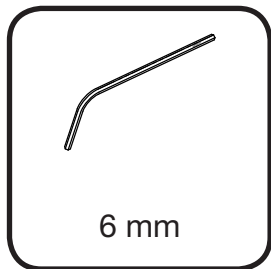

EN

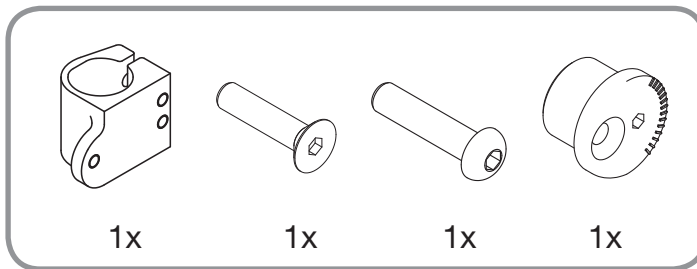

NO

1. Use a 5 mm Allen key.
Start by screwing in screw (A) followed by screw (B).
Do not tighten the screws until you have adjusted the angle of the castor wheel.
2. Insert the screw (C); do not tighten the screw until you have assembled the castor housing with fork.
3. Adjusting the angle of the castor wheel:
Place the Allen key into the rotating disk (D) and rotate to the desired angle. See the scale. Tighten screw (A), followed by screw (B).
4. When the front castor wheel is at the correct angle, the bearing housing will be in a vertical position (90°) to the base. This is achieved by using a right angle to the bearing house/floor.

1. Benytt en 5 mm umbrakonøkkel.
Skrue først inn skruen (A), deretter skruen (B). Vent med å stramme skruene til du har justert vinkelen på svinghjulet.
2. Sett inn skruen (C), men vent med å stramme skruen, til du har montert lagerhus med gaffel.
3. Justering av vinkel på svinghjul (castervinkel):
Sett umbrakonøkkelen inn i dreiskiva (D) og dreii til ønsket vinkel, se skala.
Trekk til skrue (A), deretter skrue (B).
4. Ved korrekt vinkel på svinghjulet, står lagerhuset i vertikal stilling (90°) mot underlaget. Dette oppnår du ved å benytte en rett vinkel mot lagerhus/gulv.


Recommended stud torque

Screw A = 18Nm
Screw B = 20Nm


Anbefalte tiltrekkingsmoment

Skrue A = 18Nm
Skrue B = 20Nm


1

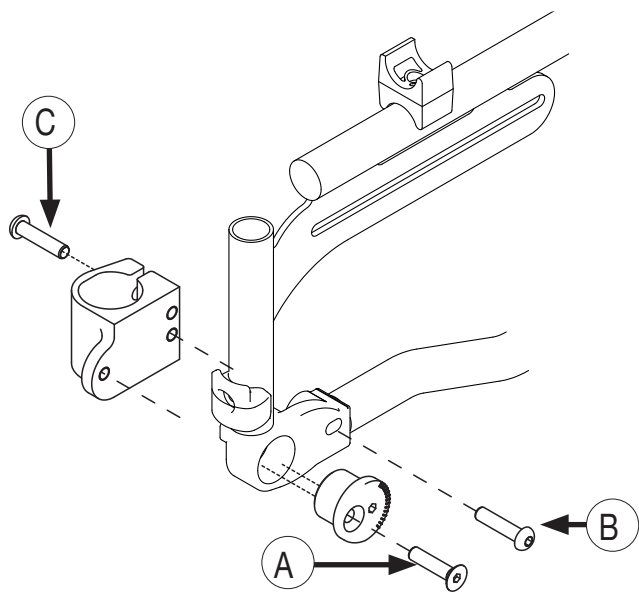

2

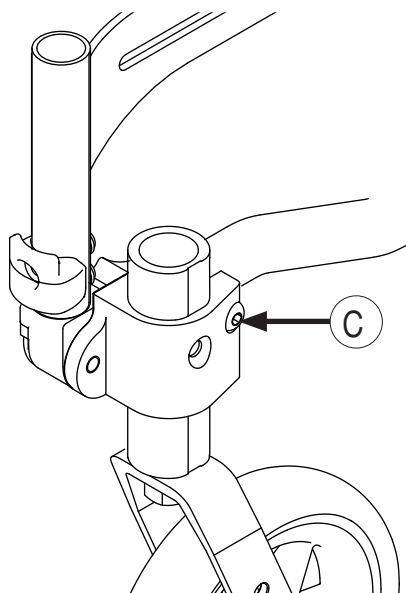

3

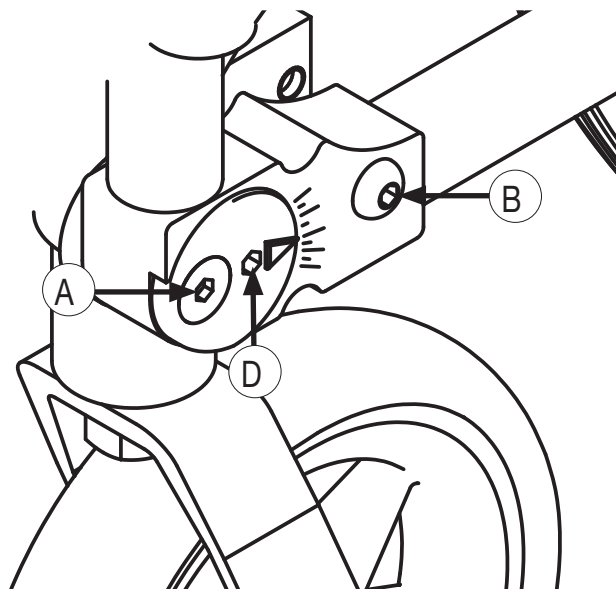



Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands

T: +31 (0)492 593 888
customerservice@sunrisemedical.nl
www.SunriseMedical.eu