


ASSEMBLY INSTRUCTION




Blockable gas spring Låsbar gassdemper

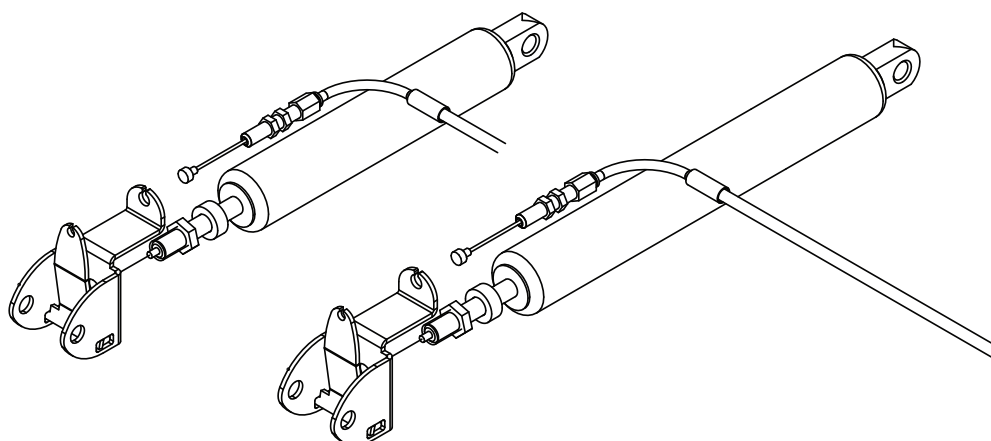

Description	Article number	
Blockable gas spring seat / Låsbar gassdemper sete	☐	3404250
Blockable gas spring back / Låsbar gassdemper rygg	☐	3405250

- **Method of changing gas spring is the same on back and seat**
- **Framgangsmåten for bytte av gassdemper for rygg og tilt er lik**



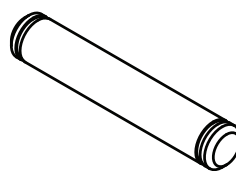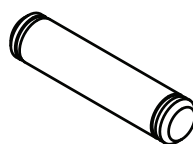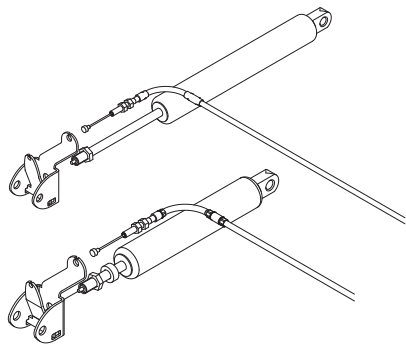

4 x sporrytter /
circlip




2 x foring / sleeve
Ø15 X Ø8 X 6mm


1 x seegringbolt /
bolt M8 x 30

1 x seegringbolt /
bolt M8 x 45

1 x mutter / nut
M8

1 x låsbar gassdemper sete eller tilt /
lockable gas spring seat or tilt

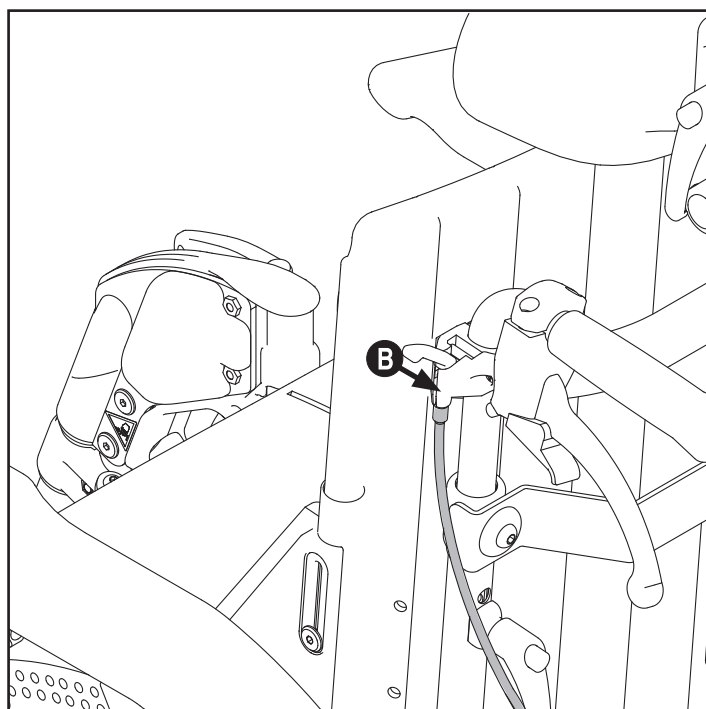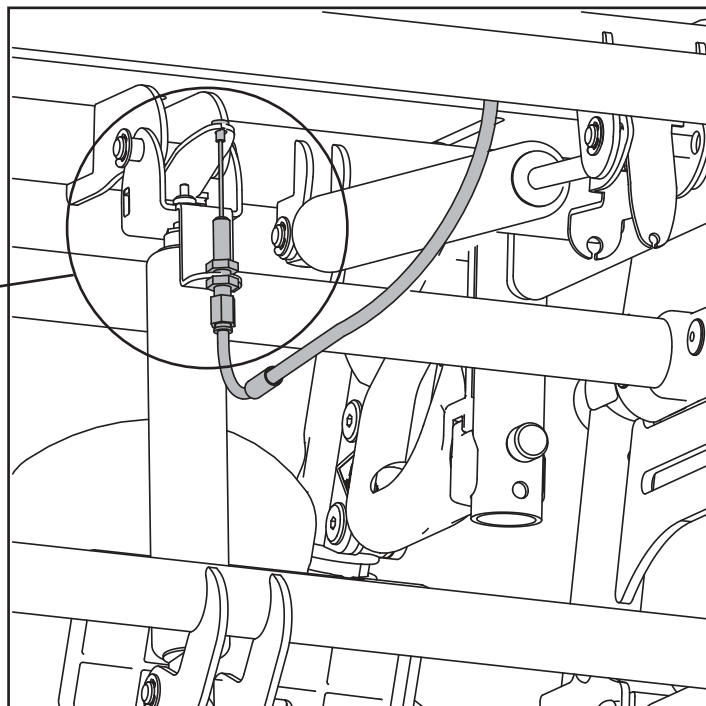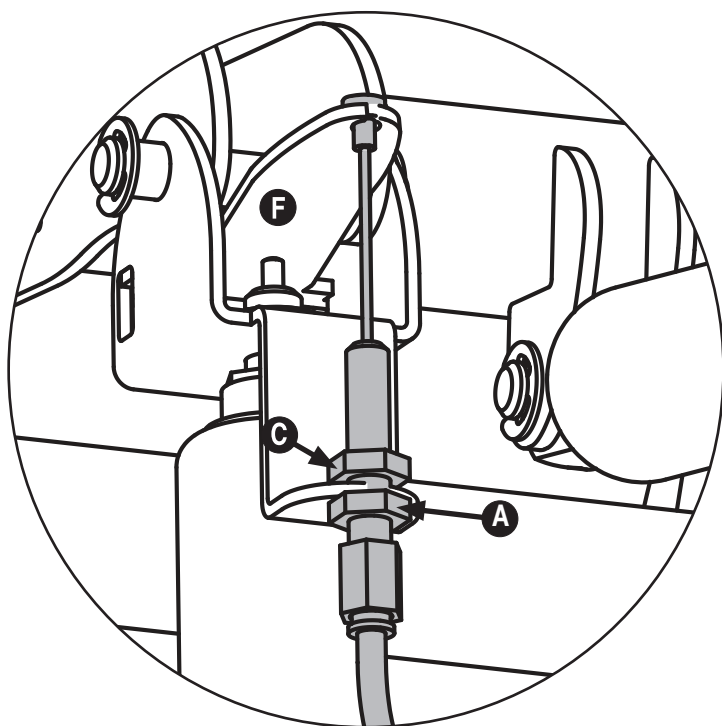

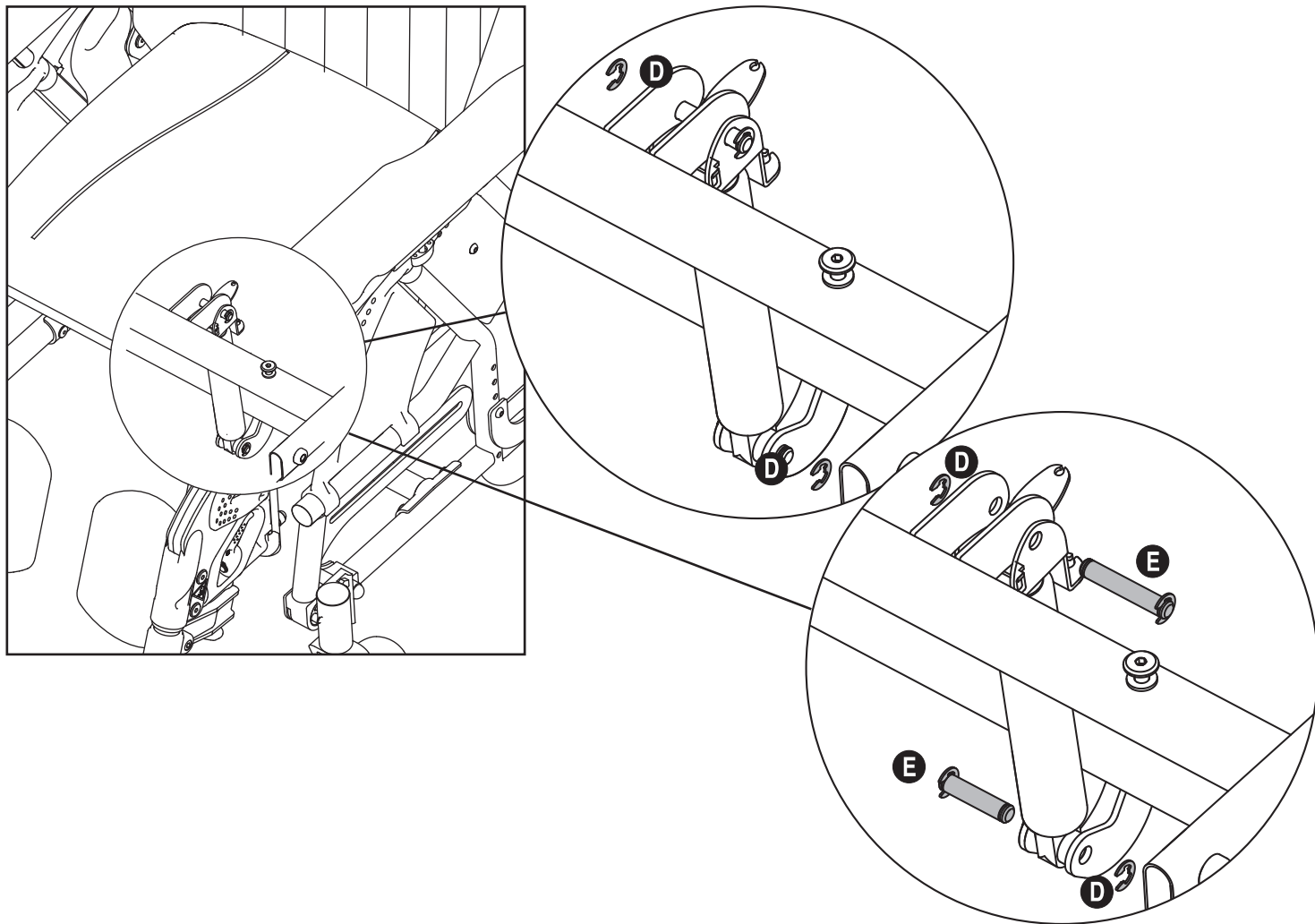

EN

Disassembling

Disassembling of the wire

- Loosen the nuts **A** as much as possible
- Remove the wire from the handle **B**
- Unscrew the nut **C** and remove the wire

Disassembling of the gas spring

- Disassemble the circlips **D** and pull the bolts out **E**

Assembling

Assembling the gas spring

- Fit the bolts **E** and the circlips **D**

Assembling the wire

- Fit the wire into the bracket **F** and into the handle **B**
- Adjust the tension of the wire by means of the nuts **A** and **C**. Tighten the nuts when the wire is correctly tightened

NO

Demontering

Demontering av vaier

- Løsne opp mutterne **A** så mye som mulig
- Ta vaieren ut av hendelen **B**
- Skru mutteren **C** helt opp og ta av vaieren

Demontering av gassdemper

- Demonter sporrytterne **D** og trekk ut boltene **E**

Montering

Montering av gassdemper

- Sett inn boltene **E** og fest sporrytterne **D**

Montering av vaier

- Plasser vaieren i braketten **F** hendelen **B**
- Juster stramming på vaieren ved hjelp av mutterne **A** og **C**. Skru til mutterne når vaieren passe stram



Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands

T: +31 (0)492 593 888
customerservice@sunrisemedical.nl
www.SunriseMedical.eu