


ASSEMBLY INSTRUCTION

Vaier for gassdemper Wire for gas spring

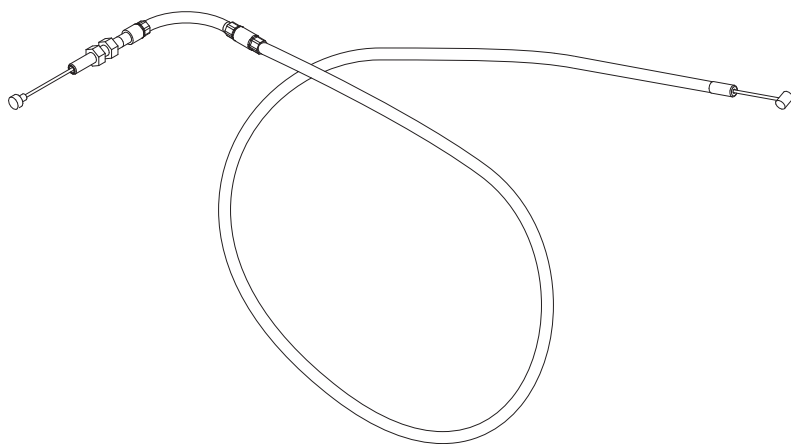

Description


Vaier for gassdemper / Wire for gas spring

Article number

3405221

- Framgangsmåten for bytte av vaier for rygg og tilt er lik
- Method of changing wire is the same on back and seat



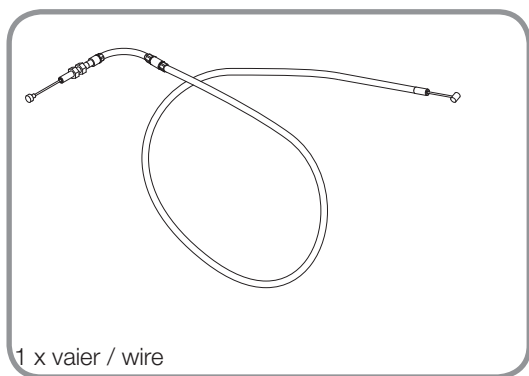

1 x vaier / wire

NO

Demontering av vaier

- Løsne opp mutterne **A** så mye som mulig
- Ta vaieren ut av hendelen **B**
- Skru mutteren **C** helt opp og ta av vaieren

Montering av vaier

- Plasser vaieren i braketten **F** hendelen **B**
- Juster stramming på vaieren ved hjelp av mutterne **A** og **C**. Skru til mutterne når vaieren passe stram




EN

Disassembling of the wire

- Loosen the nuts **A** as much as possible
- Remove the wire from the handle **B**
- Unscrew the nut **C** and remove the wire

Assembling the wire

- Fit the wire into the bracket **F** and into the handle **B**
- Adjust the tension of the wire by means of the nuts **A** and **C**. Tighten the nuts when the wire is correctly tightened

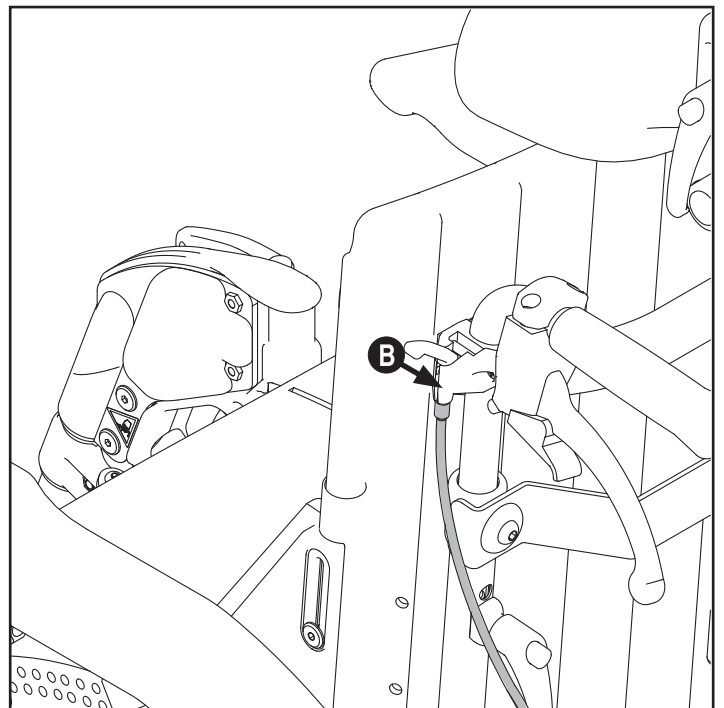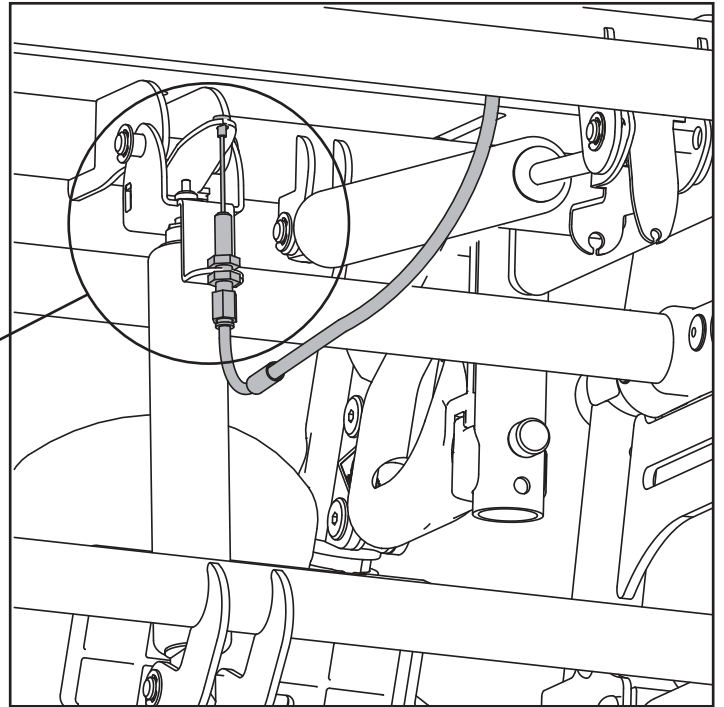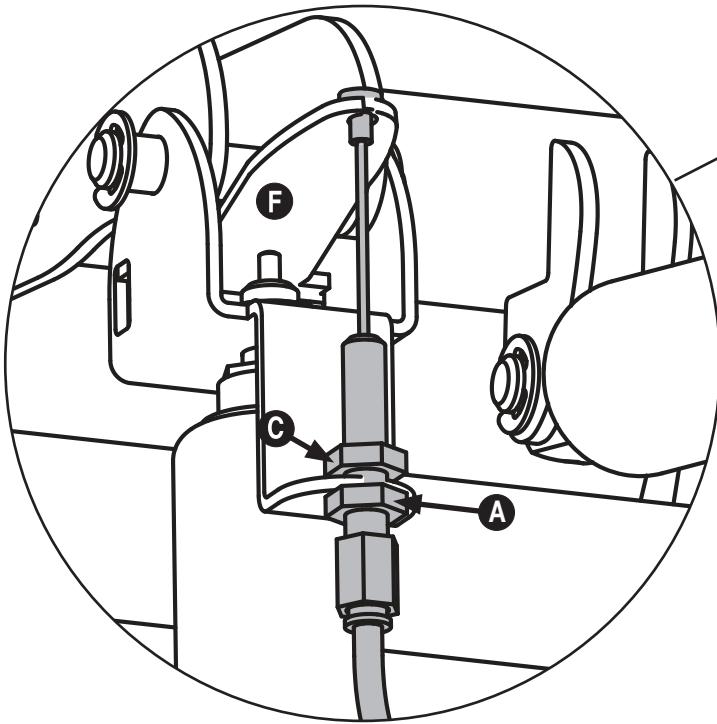



Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands

T: +31 (0)492 593 888
customerservice@sunrisemedical.nl
www.SunriseMedical.eu