


ASSEMBLY INSTRUCTION

Wire for drum brake Bremsevaier

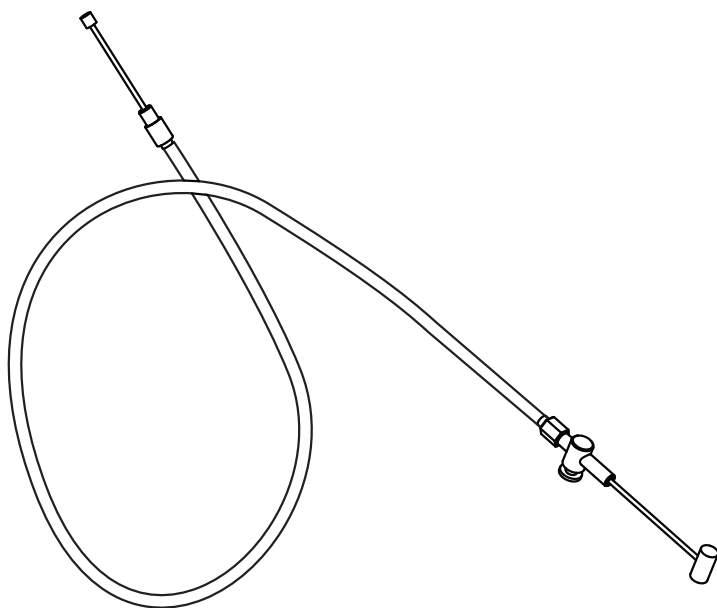


Description

Wire for drum brake / Bremsevaier for trommelbrems

Article number

3309991



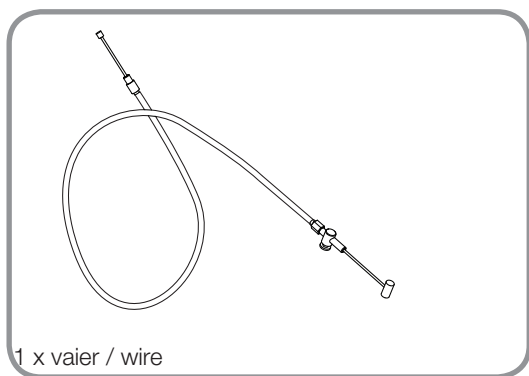

NO

Demontering av gammel bremsevaier

1. Løsne opp låsemutteren **A** så mye som mulig. Slakk vaieren ved å skru inn kabelstrammeren **B**.
Vaieren er nå løs og kan fjernes fra trommelbremsen
2. Ta bremsehåndtaket av kjørehåndtaket eller kjørebøylen ved å skru ut skruene **C**
3. Skill bremsehendelen fra hoveddelen ved å vippe den ut med en flat skrutrekker
4. Ta ut bolten **D** og trekk ut vaieren

Montering av ny bremsevaier

5. Monter den nye vaieren i bremsehendelen og monter hendelen på kjørehåndtaket eller kjørebøylen.
6. Monter den nye bremsevaieren i trommelbremsen. Fest vaierenden **E** til bremsehendelen. Skru kabelstrammeren **B** til slakkeste posisjon og fest den til sporet i ankerplata. For å justere bremsene, skru du ut kabelstrammeren **B** trinn for trinn, setter på hjulet og tester bremsen ved å snurre hjulet og klemme inn bremsehåndtaket. Hjulet skal trille lett. Stram låsemutteren **A** når bremsen er riktig justert







EN

Disassembling the old drum brake wire

1. Loosen the locking nut **A** as much as possible. Slacken the wire by screwing in the cable tightener **B**. The wire is now loose and can be removed from the drum brake.
2. Remove the brake handle from the pushing handle or pushing bar by removing the screws **C**.
3. Separate the brake handle from the brake-handle mounting by levering it out with a flat screwdriver.
4. Remove the bolt **D** and pull the wire out.

Assembling the new drum brake wire

5. Fix the new wire to the brake handle and attach to the pushing handle or the pushing handle.
6. Attach the new brake wire to the drum brake. Fix the wire end **E** to the brake handle. Turn the cable tightener **B** to its slackest position and fix it to the track in the anchor plate. To adjust the brakes, unscrew the cable tightener **B** step by step, attach the wheel and test the brake by turning the wheel and squeezing the brake handle. The wheel should roll freely. Tighten the locking nut **A** when the brake has been adjusted correctly.

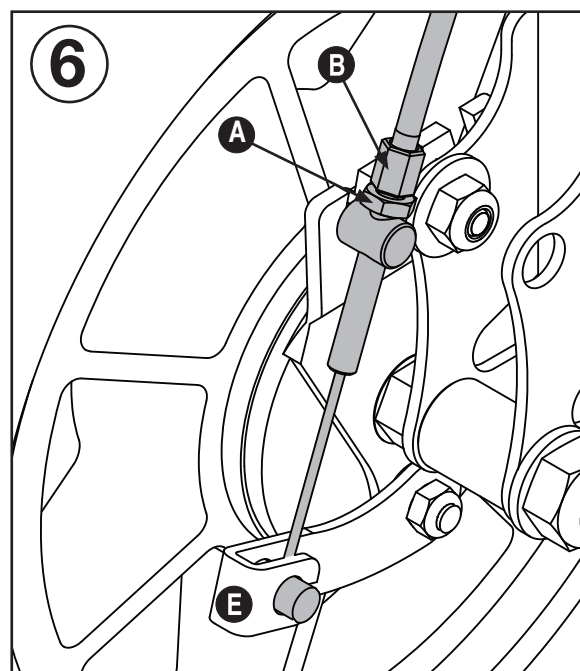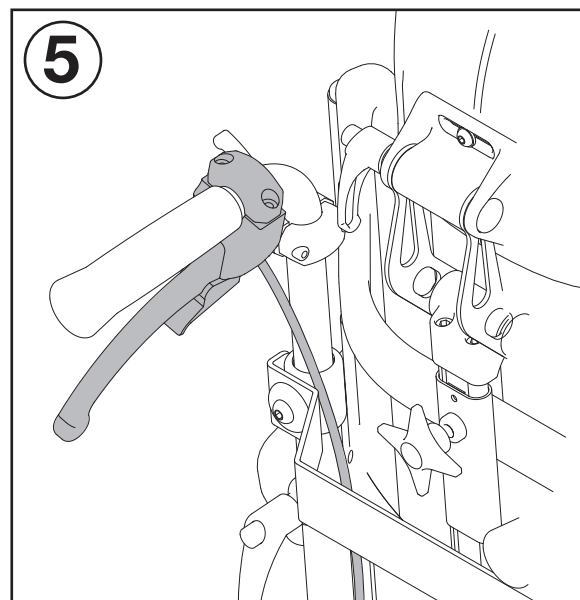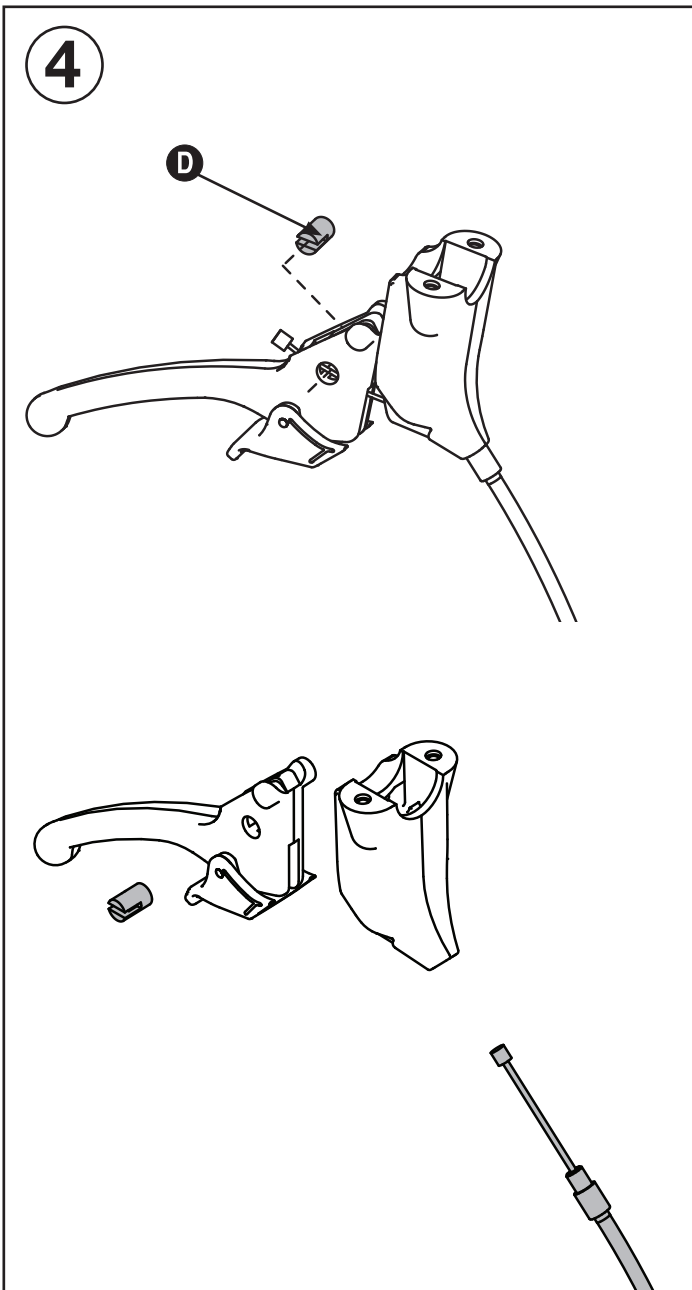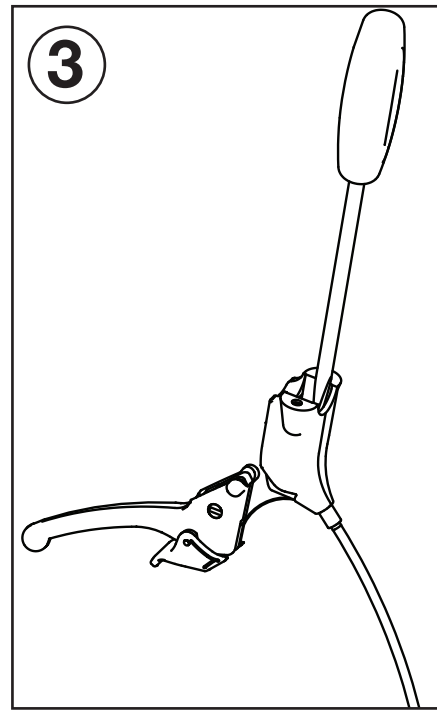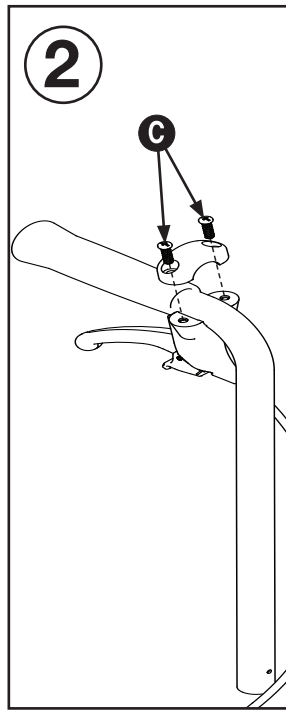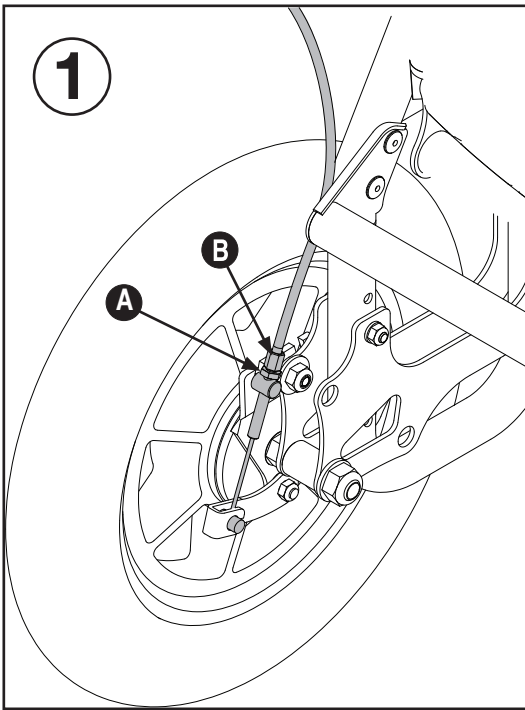



Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands

T: +31 (0)492 593 888
customerservice@sunrisemedical.nl
www.SunriseMedical.eu