

ASSEMBLY INSTRUCTION

External Joystick / External Joystick Lite



Description	Article number	
External Joystick - Fixed mounting on worktop	☐	1016499
External Joystick - Manual flip-down mounting on worktop	☐	1016500
External Joystick Lite - Fixed mounting on worktop	☐	1016505
External Joystick Lite - Manual flip-down mounting on worktop	☐	1016506






Included in kit 00355.0477 00355.0471 1015013 1015014 1016726 1016727 1016716

1016499	1	1	1		1		1
1016500	1	1	1			1	1
1016505	1	1		1	1		1
1016506	1	1		1		1	1



Torx 30 mm

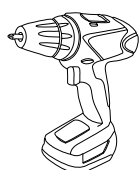

Dril

EN

1. Fit the worktop bracket to the armrest. Slide the worktop tube into the bracket and tighten the locking knob.
2. Assemble the Omni screen bracket.
3. Assemble the swing-away controller bracket.
4. Drill holes in the worktop for the buddy buttons (to be done with user in the wheelchair). Mount the controller to the worktop and set up the cabling.

In order to set up the system, see R-net Omni technical manual.

NO



1. Monter bordbraketten på armlenet. Skyv bordrøret inn i braketten og stram til låserattet.
2. Monter Omni-skjermbraketten
3. Monter den utsvingbare styreboksbraketten.
4. Bor hull i bordplaten til Buddy-knappene (gjøres med brukeren i rullestolen). Monter styreboksen på bordplaten og sett opp kablingen.

Se den tekniske manualen for R-net Omni for oppsett av systemet.

SE

1. Montera bordsfästet på armstödet. Skjut in bordsröret på fästet och dra åt låsknappen.
2. Montera Omni-skärmens fäste.
3. Montera undansvingbara controller fästet.
4. Borra hål i bordet för buddy-kontakterna (görs med brukaren i rullstolen). Montera controller på bordet och installera kablaget.

För att installera systemet, se R-net Omni teknisk manual.

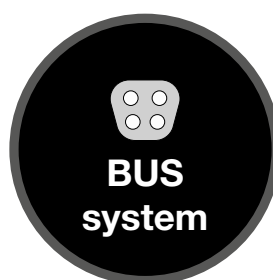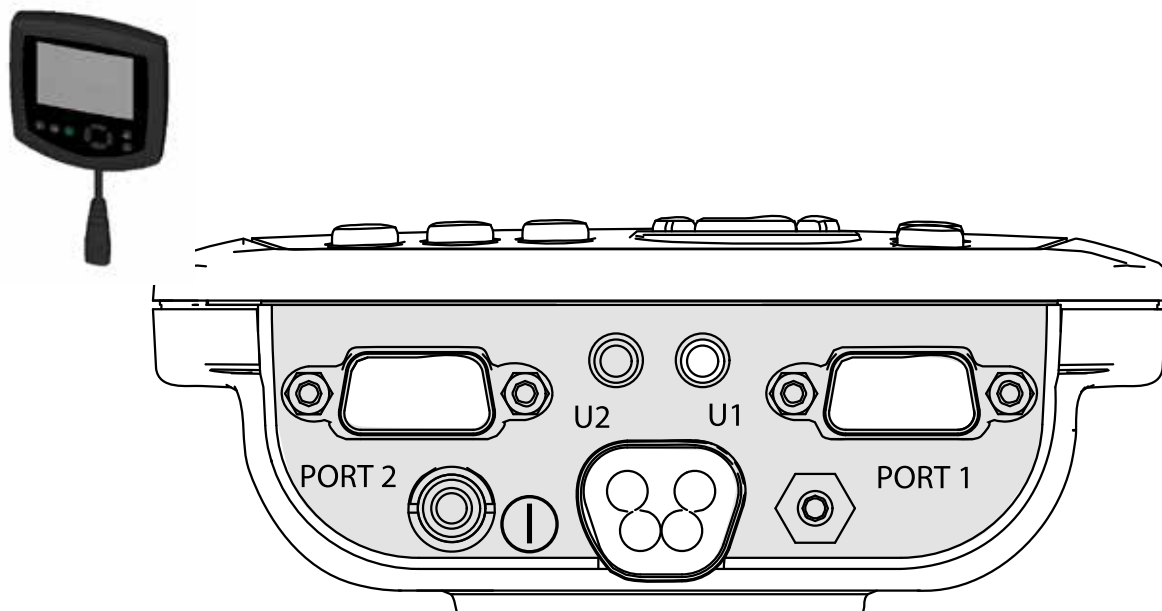

Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands

T: +31 (0)492 593 888
customerservice@sunrisemedical.nl
www.SunriseMedical.eu