

ASSEMBLY INSTRUCTION

External Joystick / External Joystick Lite



Description

Article number

External Joystick - Mounting on manual swing-away chin arm



1016502

External Joystick Lite - Mounting on manual swing-away chin arm



1016508



Included in kit

1015013



1015014



1016719



1016497



1016856



9010243*



1015019



1016716

1016502

1

1

1

1

1

1

1

1016508

1

1

1

1

1

1

1



Torx 30 mm

EN

1. Assemble the bracket for the chin arm.*
2. Slide the chin arm into the back tube.
3. Assemble the Omni screen bracket.
4. Assemble the swing-away controller bracket.
5. Set up the cabling.

* Only in combination with Sedeo Pro+ back.

In order to set up the system, see R-net Omni technical manual.

NO

1. Monter braketten for hakestyringsarmen.*
2. Skyv armen inn i ryggørret.
3. Monter Omni-skjermbraketten
4. Monter den utsvingbare styreboksbraketten.
5. Sett opp kablene.

* Kun i kombinasjon med Sedeo Pro+ rygg.


Se den tekniske manualen for R-net Omni for oppsett av systemet.

SE

1. Montera fästet för hakstyrning.*
2. Skjut in hakstyrningsarmen i ryggörret.
3. Montera Omni-skärmens fäste.
4. Montera undansvingbara controller fästet.
5. Installera kablaget.

* Endast i kombination med Sedeo Pro+ rygg.

För att installera systemet, se R-net Omni teknisk manual.

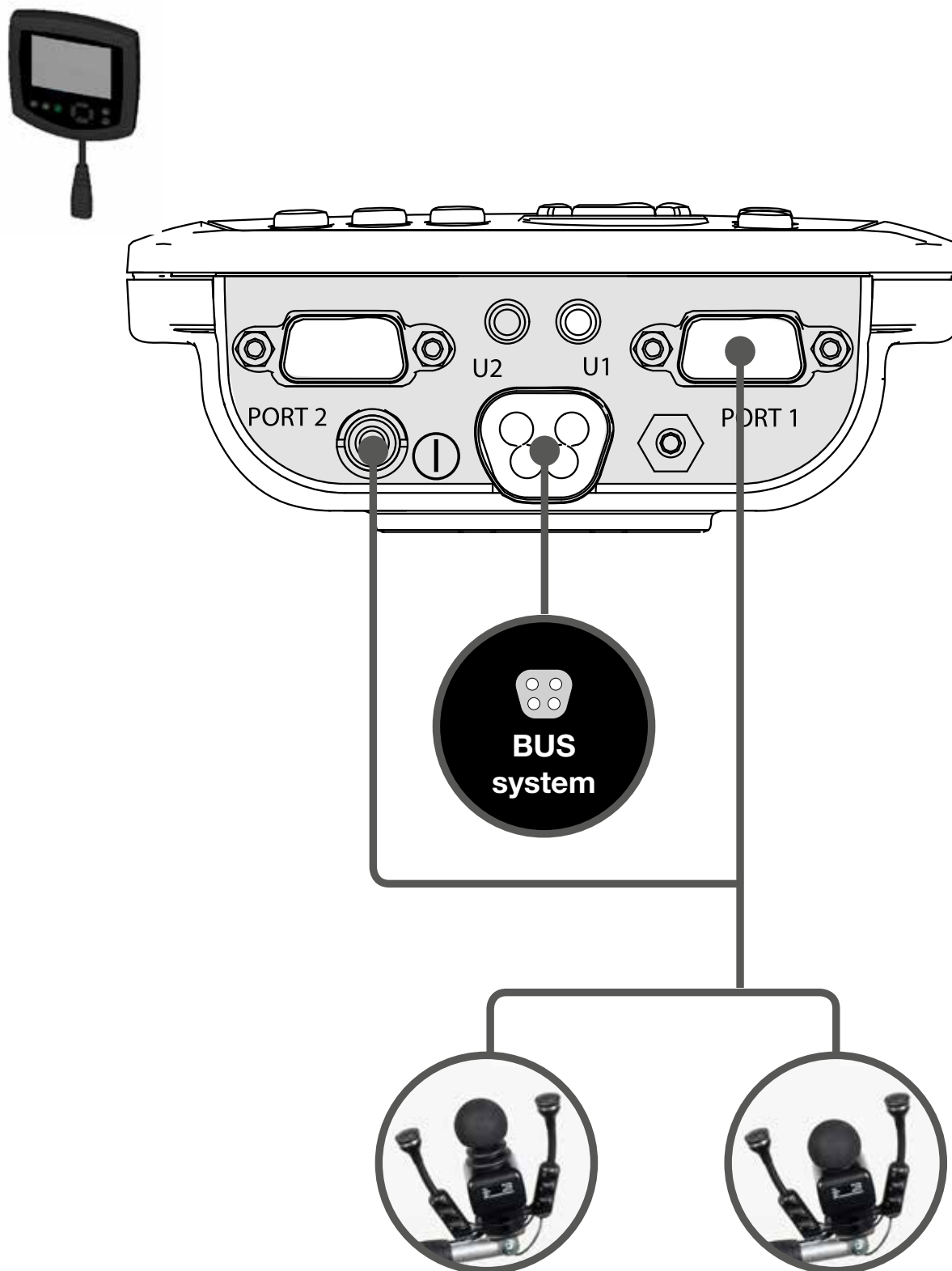

Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands

T: +31 (0)492 593 888
customerservice@sunrisemedical.nl
www.SunriseMedical.eu