

ASSEMBLY INSTRUCTION

Mini Flex Switch Joystick / Micro Pilot Joystick / Micro Guide Joystick



Description

Article number

Mini Flex Switch Joystick - Mounting on electrical swing-away chin arm



1016516

Micro Pilot Joystick - Mounting on electrical swing-away chin arm



1016514

Micro Guide Joystick - Mounting on electrical swing-away chin arm



1016674





Included in kit

00355.0472 00355.0471 00355.0477 00355.0448 1016679 1016681 1016498 1016720 1016722 1016856 9010243* 1016717 1016716

1016516	1	1		1		2	1		1	1	2	1
1016514	1	1	1		1	2		1	1	1	3	1
1016674	1	1	1			2		1	1	1	3	1



Torx 30 mm

EN

1. Assemble the bracket for the chin arm.*
2. Slide the chin arm into the back tube.
3. Assemble the brackets for buddy buttons on headrest.
4. Assemble the Omni screen bracket.
5. Assemble the swing-away controller bracket.
6. Set up the cabling.

* Only in combination with Sedeo Pro+ back.

In order to set up the system, see R-net Omni technical manual.

NO

1. Monter braketten for hakestyringsarmen.*
2. Skyv hakestyringsarmen inn i ryggørret.
3. Monter brakettene til Buddy-knappene på hodestøtten.
4. Monter Omni-skjermbraketten
5. Monter den utsvingbare styreboksbraketten.
6. Sett opp kablene.

* Kun i kombinasjon med Sedeo Pro+ rygg.




Se den tekniske håndboken for R-net Omni for oppsett av systemet.


SE

1. Montera fästet för hakstyrning.*
2. Skjut in hakstyrningsarmen i ryggörret.
3. Montera fäste för buddy-kontakter på nackstöd.
4. Montera Omni-skärmens fäste.
5. Montera undansvängbara controller fästet.
6. Installera kablagen.

* Endast i kombination med Sedeo Pro+ rygg.

För att installera systemet, se R-net Omni teknisk manual.

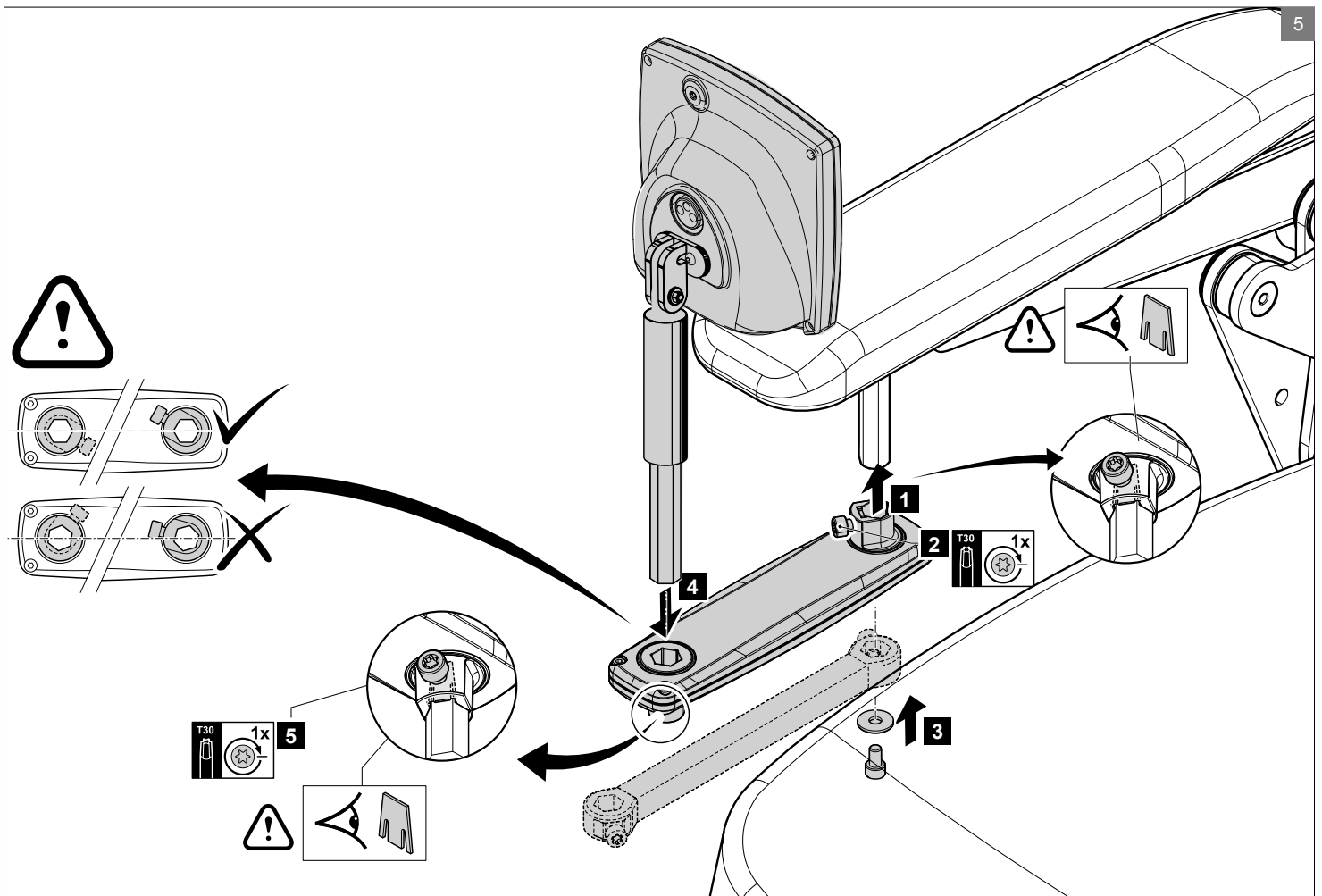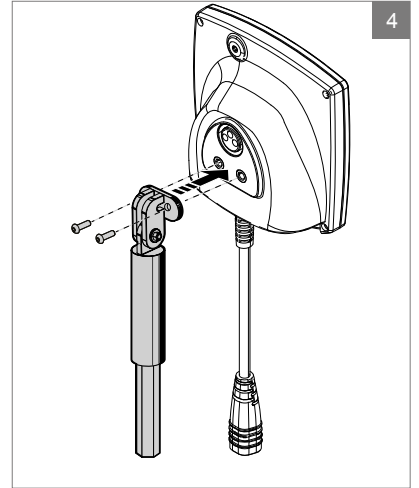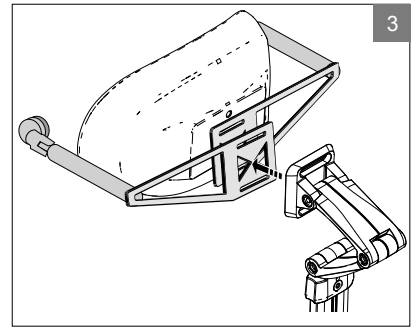

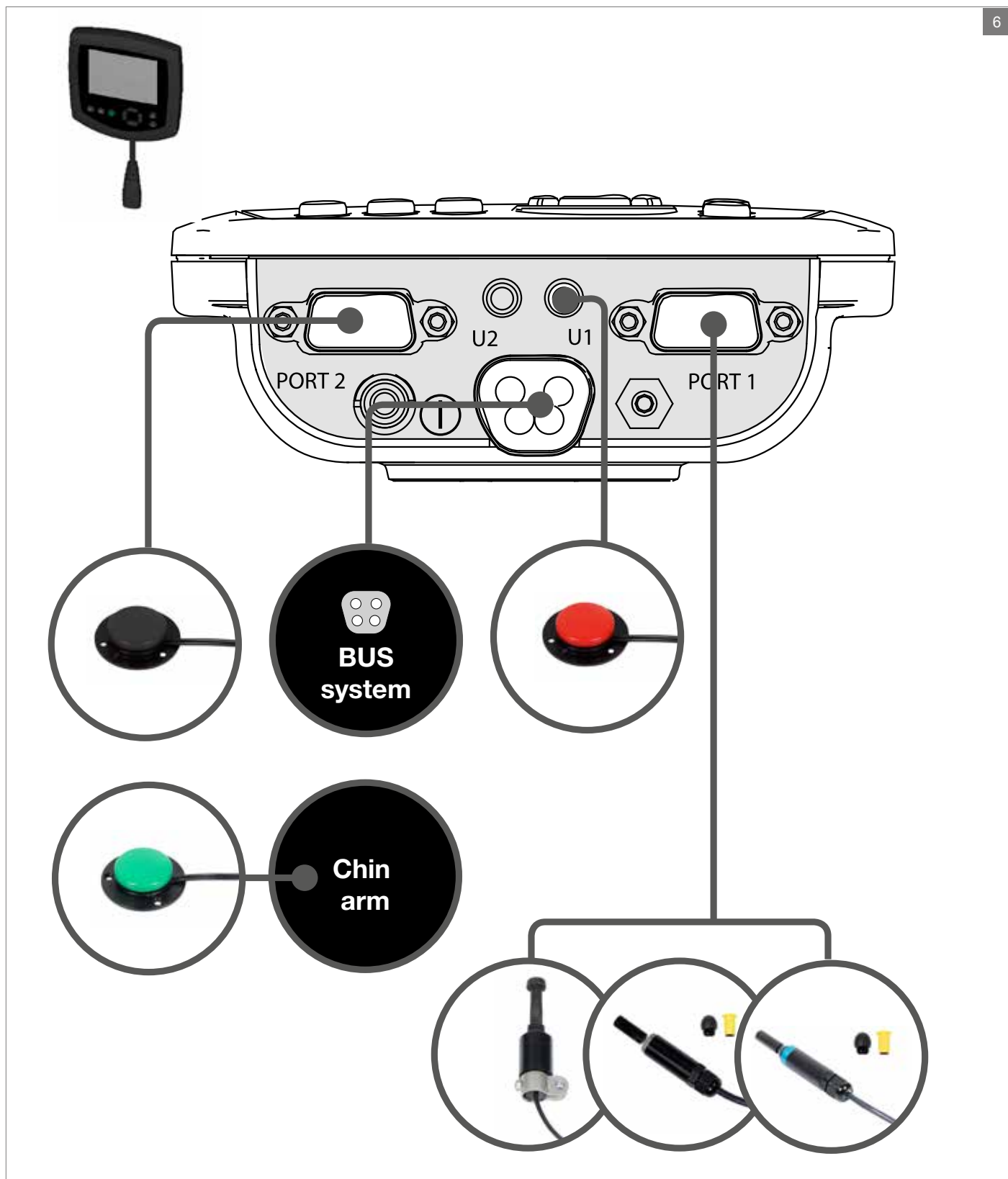

Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands

T: +31 (0)492 593 888
customerservice@sunrisemedical.nl
www.SunriseMedical.eu