

ASSEMBLY INSTRUCTION

Proximity Head Control Array



Description

Proximity Head Control Array - Mounting on backrest

Article number



1016671



Included in kit

1016671



1016678

1



00355.0477

1



00355.0471

1



1016717

2

1016676

1



1016492

2



1016716

1



Torx 30 mm

EN

1. Slide the Proximity head array into fixation on the backrest.
2. Assemble the buddy buttons on the headrest.
3. Assemble the Omni screen bracket.
4. Assemble the swing-away controller bracket.
5. Set up the cabling.

 In order to set up the system, see R-net Omni technical manual.

NO




1. Monter hodestyringen i festet på ryggstøtten.
2. Monter Buddy-knappene på nakkestøtten.
3. Monter Omni-skjermbraketten
4. Monter den utsvingbare styreboksbraketten.
5. Sett opp kablene.


 Se den tekniske manualen for R-net Omni for oppsett av systemet.

SE

1. Skjut huvudstyrningen med kontakter för fixering mot ryggstödet
2. Montera buddy-kontakterna på nackstöd.
3. Montera Omni-skärmens fäste.
4. Montera undansvingbara controller fästet.
5. Installera kablaget.

 För att installera systemet, se R-net Omni teknisk manual.

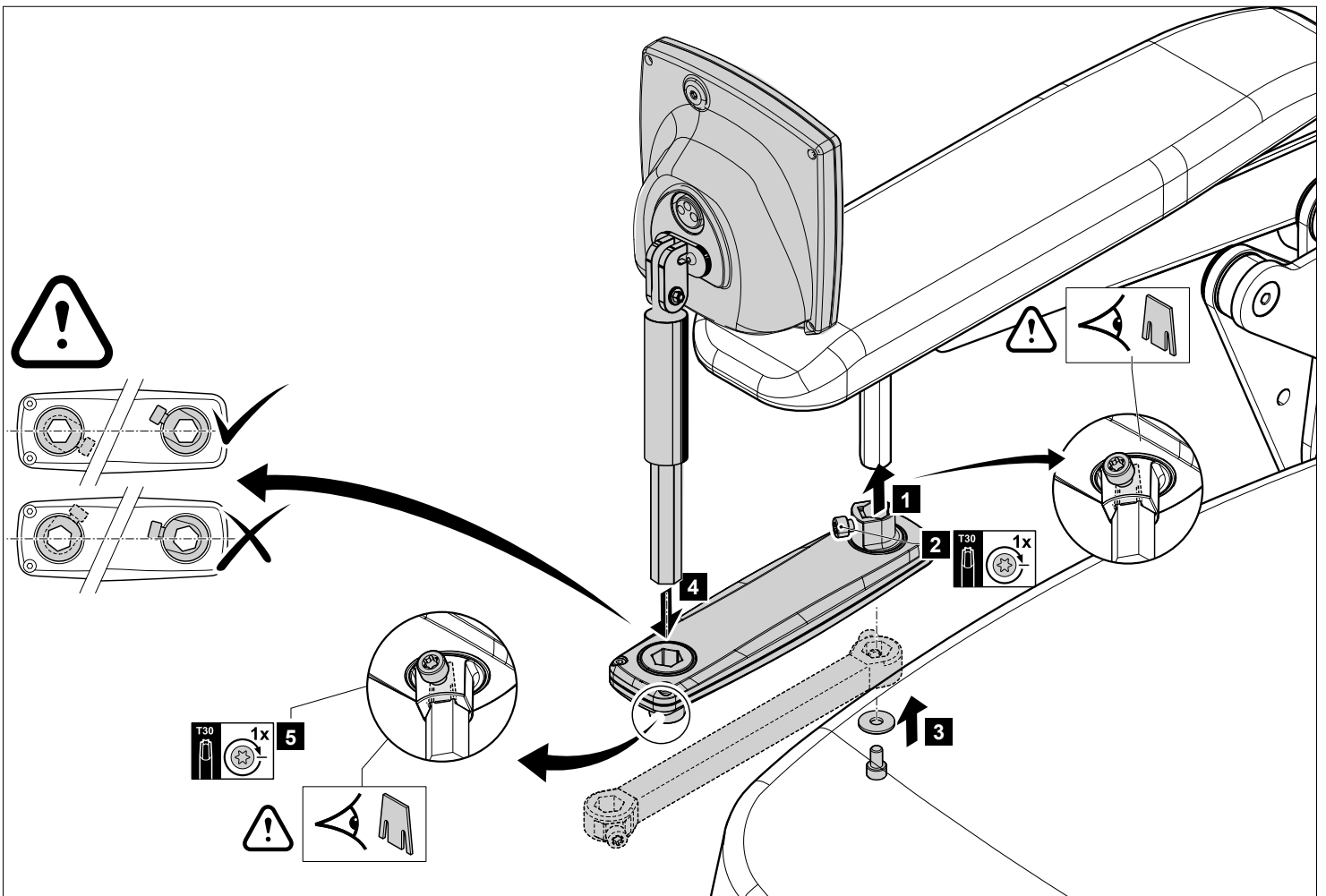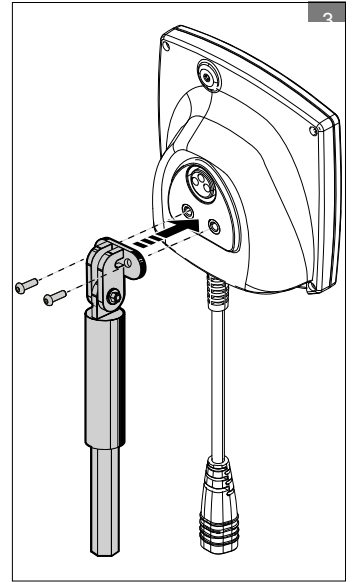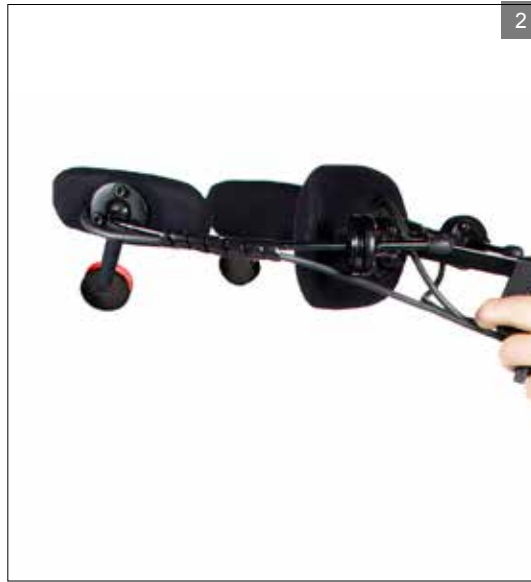

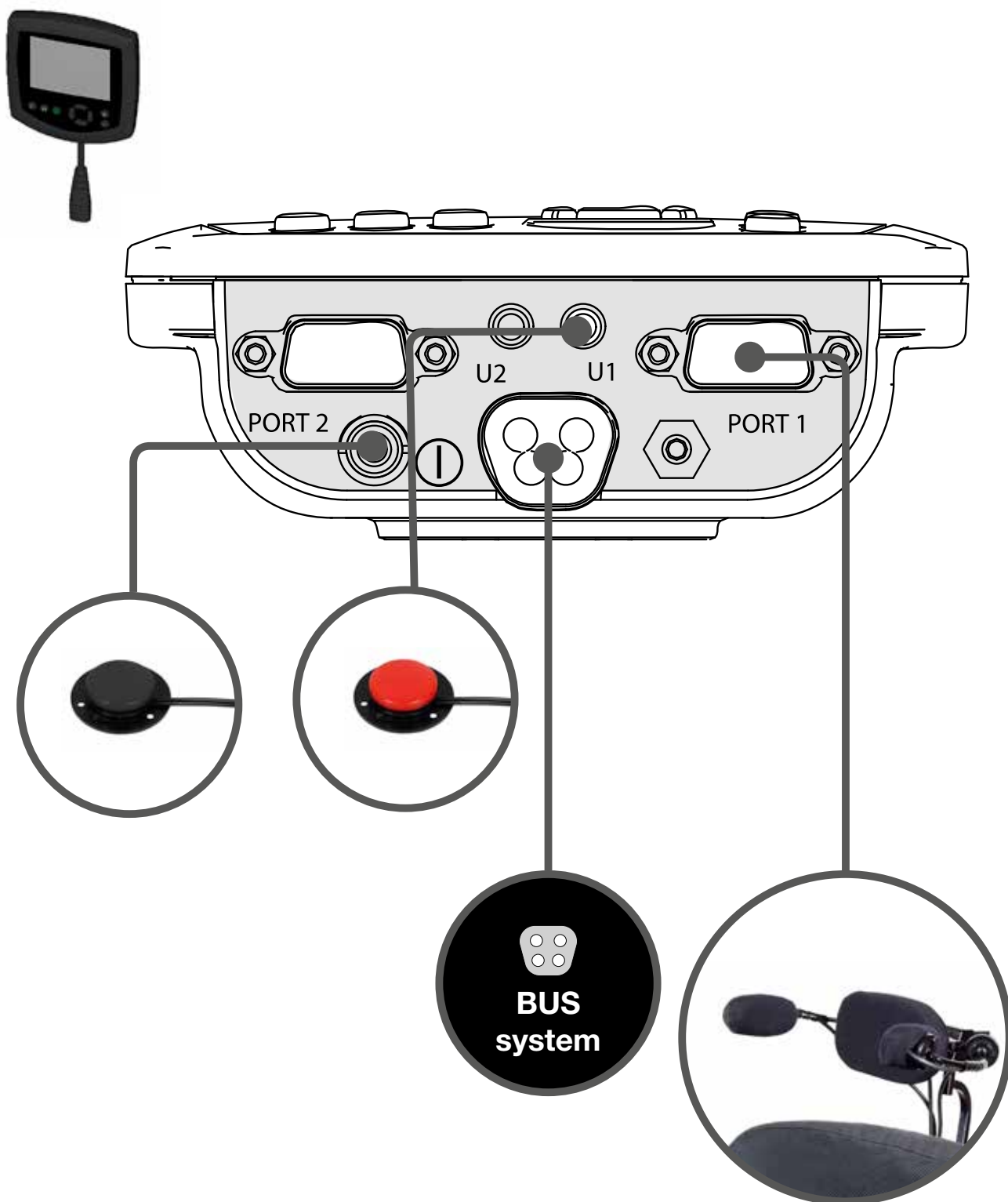

Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands

T: +31 (0)492 593 888
customerservice@sunrisemedical.nl
www.SunriseMedical.eu