JAY®

JAY® Fit Back

Important Consumer Information

NOTE: This manual contains important instructions that must be passed on to the user of this product.

Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

WARRANTY REGISTRATION: To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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
www.SunriseMedical.com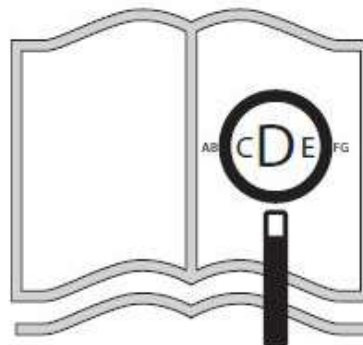

EN

If you are visually impaired, this document can be viewed in PDF format at **www.SunriseMedical.co.uk**

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**ENGLISH**

INTRODUCTION	4
JAY FIT BACK	4
GENERAL WARNINGS	5
USE AND MAINTENANCE	6
INSTALLATION AND ADJUSTMENT	9
ACCESSORIES	11
WARRANTY	18
	19

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SUNRISE LISTENS:

Thank you for choosing a JAY® Fit Back. We want to hear your questions or comments about this manual, the safety and reliability of your back system and the service you receive from your Sunrise supplier.

Please feel free to write or call us at the address and telephone number below:

Sunrise Medical Ltd.
Thorns Road
Brierley Hill
West Midlands
DY5 2LD
Phone: 0845 605 66 88
Fax: 0845 605 66 89
www.SunriseMedical.co.uk

Be sure to maintain in a safe place all the instructions and manuals you received with your back system. They contain important user and safety information and should be referred to often.

FOR ANSWERS TO YOUR QUESTIONS:

Your authorised supplier knows your back system best, and can answer most of your questions about safety, use and maintenance and important adjustments. For future reference, fill in the following:

Supplier:
Address:
.....
Telephone:
Serial Number:
Date of Purchase:

INTRODUCTION**⚠ WARNING!****READ BEFORE USE:**

Failure to follow these recommendations and instructions can cause equipment failures that result in serious injury.

CLINICAL CONSULTATION

Sunrise Medical recommends that a clinician, such as a doctor or therapist, experienced in back and positioning be consulted to determine if a Jay Fit Back is appropriate.

HARDWARE COMPATIBILITY

Jay Fit Backs are designed to be physically compatible with most types of wheelchairs. Backs should only be installed by an Authorised Sunrise Medical Supplier. To determine hardware compatibility for a particular wheelchair please follow the instructions in section II, "Manual Wheelchair Back Cane Compatibility". Sunrise Medical manufactures two styles of mounting hardware, Standard and Quick-Mount. Standard hardware is not recommended for reclining wheelchairs, standing wheelchairs and wheelchairs that tilt in space beyond 30 degrees.

HARDWARE MAINTENANCE

Sunrise recommends that all fasteners on Jay Fit Backs be checked for wear, such as loose bolts or broken components every 6 months. Loose fasteners should be re-tightened according to the installation instructions. All fasteners should be tightened to 2.3 Nm (20 in.-lbs.). Contact your authorised Sunrise Medical Supplier immediately to replace any broken components. Do not continue to use the back after identifying loose or broken components.

MOTOR VEHICLE TRANSPORTATION

If you have any questions regarding these instructions contact your Authorised Sunrise Medical Supplier or Sunrise Medical directly.

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in back and positioning be consulted to determine if the JAY® Fit back is an appropriate solution. These back systems should only be installed by an Authorised Sunrise Medical dealer.

JAY® FIT BACK (FIG.1.0-2.0)


The JAY®Fit Back system is designed to provide the important benefits of a proper back support for a wide variety of users. It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is light-weight, durable and easy to upgrade.

The intended lifetime of this product is 5 years.

For further information on the full specification and options and accessories please refer to the order form.

All information is subject to change without notification. Please consult Sunrise Medical with any queries you may have.

Fig.1.0

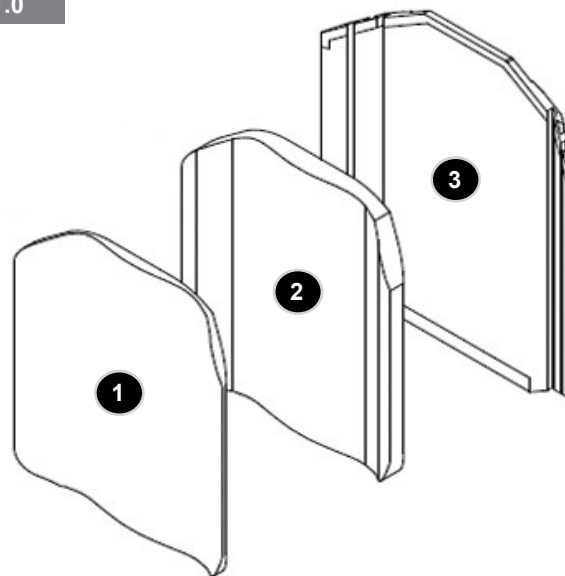**FIG.1.0.
JAY FIT BACK SUPPORT**


1. Cover.
2. Foam.
3. Shell.

**FIG.2.0.
JAY FIT BACK HARDWARE ADJUSTABLE
QUICK MOUNT**

1. Push Handle Collar.
2. Securing Hardware.
3. Channel Nuts.
4. Back Shell.

- A. Attachment Brackets.
B. Snaptide Hardware.

Fig.2.0

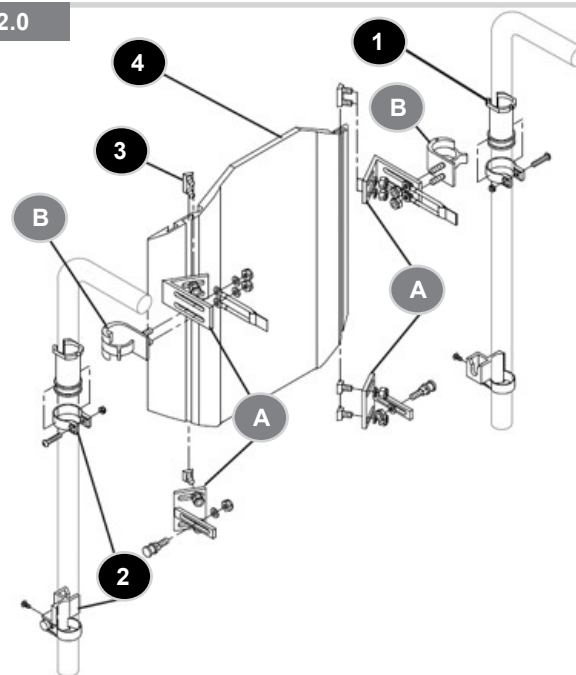Word	Definition
 DANGER!	Advice to the user of potential risk of serious injury or death if the advice is not followed
 WARNING!	Advice to the user of a risk of injury if the advice is not followed
 CAUTION!	Advice to user that potential damage to equipment may occur if the advice is not followed
NOTE:	General advice or best practice
	Reference To Additional Documentation

A. WEIGHT LIMITS

DANGER!

- NEVER exceed the weight limit specified by Sunrise Medical.
- The weight capacity provided is for the combined weight of a rider and items carried using on-board storage.
- If you do exceed the weight limit, damage to your product, a fall, tip over or loss of control may occur and cause severe injury to the rider or others.

Weight Capacities:

JAY Fit Back - 113 kg, (250lbs).

B. INTENDED USE

JAY Fit back is intended use is to provide support and comfort to persons limited to a seated position.

DANGER!

- DO NOT use this system for purposes other than what is intended by the manufacturer.
- If you fail to heed these warnings damage to your wheelchair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

C. ATTENDANTS AND CAREGIVERS

WARNING!

- Before you assist a rider, be sure to read all warnings contained in this manual, and follow all instructions that apply.
- Be aware that after consulting a healthcare advisor, you will need to learn safe and proven body mechanics to use and create assistive methods best suited to your abilities.

D. ACCESSORIES

WARNING!

- Unauthorised modifications or use of parts, or accessories not supplied or approved by Sunrise Medical may change the wheelchair structure.
- This will void the warranty and may cause a safety hazard.

Example:

Some problems that may occur, but are not limited to:

1. Adding a component to the frame, changing the structural integrity of the wheelchair.
2. Any modification or disassembly can potentially create an unsafe situation where rider and/or attendant are put at risk.

E. REDUCE THE RISK OF AN ACCIDENT

WARNING!

1. BEFORE riding, you should be trained in the safe use of this wheelchair by your health care advisor.
2. Practice bending, reaching and transfers until you know the limit of your ability. Have someone help you until you know what can cause a fall or tip-over and how to avoid doing so.
3. Be aware that you must develop your own methods for safe use best suited to your level of function and ability.
4. NEVER try a new manoeuvre on your own until you are sure you can do it safely.
5. Get to know the areas where you plan to use your wheelchair. Look for hazards and learn how to avoid them.

F. SAFETY CHECKLIST**⚠ DANGER!**

Before Each Use Of Your Back system:

1. Make sure the attachment (mounting hardware is secure and tight. Check for noise, vibration, or a change in ease of use. (may cause loose fasteners, or damage to your wheelchair).
2. Immediately repair any problem.
3. Your Authorised supplier can help you find and correct the problem.
4. If your wheelchair has anti-tip tubes, lock them in place.
5. If you fail to heed these warnings, damage to your wheelchair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

G. CHANGES & ADJUSTMENTS**⚠ DANGER!**

- See your Authorised supplier and have them adjust back components any time a change or adjustment needs to be made.
- Unauthorised modifications, or use of parts not supplied or approved by Sunrise may change the wheelchair structure. This will void the warranty and may cause a safety hazard.
- If you fail to heed these warnings, damage to your wheelchair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

H. CENTRE OF GRAVITY**⚠ WARNING!**

- Installing a back support on a wheelchair may affect the centre of gravity of the wheelchair, and may cause the wheelchair to tip back-wards potentially resulting in injury to the rider.

I. ANTI-TIPS**⚠ WARNING!**

Anti-tip tubes can help keep your wheelchair from tipping over backward in normal conditions.

- Sunrise recommends the use of anti-tip tubes
- ALWAYS assess the need for anti-tips that could be added to the wheelchair to help increase the stability of the wheelchair and back system.

J. PUSHING OR LIFTING**⚠ DANGER!**

- Do not use the JAY Fit back to move or push the wheelchair.
- Do not use the back handle to lift the wheelchair. It is intended only for Back removal.
- Improper use of the back by pushing or pulling may cause the back to unexpectedly detach from the wheelchair.
- If you fail to heed these warnings damage to your wheelchair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

K. COMPATIBILITY**⚠ DANGER!**

The JAY Fit is designed to be compatible with most wheelchairs with the following exception:

- Wheelchairs that are angle adjustable with recline or tilt, and result in a back angle greater than 60° from the vertical.
- The JAY Fit Back used with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair. This could result in a fall or potential injury of the rider and/or caregiver.

L. POSITIONING BELTS (OPTIONAL)**⚠ DANGER!**

Use positioning belts ONLY to help support the rider's posture. Improper use of these belts may cause severe injury to or death to the rider.

1. Make sure the rider does not slide down in the wheelchair seat. If this occurs, the rider may suffer chest compression or suffocate due to pressure from the belts.
2. The belts must be snug, but must not be so tight that they interfere with breathing. You should be able to slide your open hand, flat, between the belt and the rider.
3. A pelvic wedge or a similar device can help keep the rider from sliding down in the seat. Consult with the rider's doctor, nurse or therapist to find out if the rider needs such a device.
4. Use positioning belts only with a rider who can co-operate. Make sure the rider can easily remove the belts in an emergency.
5. NEVER Use Positioning Belts:
 - a. As a patient restraint. A restraint requires a doctor's order.
 - b. On a rider who is comatose or agitated.
 - c. As a motor vehicle restraint. In an accident or sudden stop the rider may be thrown from the wheelchair. Wheelchair seat belts will not prevent this, and further injury may result from the belts or straps.
- If you fail to heed these warnings damage to your wheelchair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

M. CUSHIONS**⚠ WARNING!**

- If you suffer from pressure sores or if you are at risk that they will occur, you may need a special seat system or a device to control your posture.
- Consult your doctor, nurse or therapist to find out if you need such a device for your well-being.

N. FASTENERS**⚠ DANGER!**

Many of the screws, bolts and nuts on this wheelchair are special high-strength fasteners.

Use of improper fasteners may cause your wheelchair to fail.

- ONLY use fasteners provided by an authorised supplier (or ones of the same type and strength, as indicated by the markings on the heads).
- Over or under-tightened fasteners may fail or cause damage to wheelchair parts.
- If bolts or screws become loose, tighten them as soon as you can.
- If you fail to heed these warnings damage to your wheelchair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

O. PINCH POINTS**⚠ WARNING!**

When hinged or adjustable components are moved or adjusted, there is a potential for a pinch point to exist.

To avoid pinch points:

- Use an open Palm to lock or fasten hardware and components.
- Always be aware of pinch point areas and use caution.

P. POSTURAL SUPPORT ACCESSORIES**⚠ DANGER!**

When using Removable, adjustable, or hinged components:

1. Check the function of all moveable component accessories that may be added to this device.
 2. Ensure that these items are fully adjusted and locked into position to provide the necessary support to the user.
 3. Items such as swing-away laterals, abductors, and head supports are an important to the safety and health of the user and must be checked regularly to make sure they have not moved or been left unlocked accidentally.
- If you fail to heed these warnings damage to your wheelchair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Q. ENVIRONMENTAL CONDITIONS**⚠ CAUTION!**

Contact with water or excess moisture may cause your wheelchair to rust or corrode. Avoid all extreme weather situations if possible.

- Do not use your components in a shower, pool or other body of water. The parts are not water-tight and may rust or corrode from the inside.
- Avoid excess moisture (for example, do not leave your wheelchair in a damp bathroom while taking a shower).
- Dry your components as soon as you can if they get wet, or if you use water to clean them.

R. JAY FIT BACK TRANSIT SAFETY**⚠ DANGER!**

JAY Fit back has been dynamically tested for use in a motor vehicle. This system conforms with ANSI/RESNA-WC 19 and ISO Standard 7176-19. If JAY Fit back is to be used in a transit situation, it must be mounted to a wheelchair that conforms to the performance requirements of ANSI/RESNA-WC 19 and ISO Standard 7176-19. For Rider safety, Please follow all installation, use, and maintenance instructions within this manual as well as the transit instructions listed below:

- If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the vehicle restraints.
- The distance between the top of the user's shoulder and the top of the back should not exceed 6.5" (16.5 cm).
- The wheelchair must be labelled as appropriate for use as a seat in a motor vehicle, dynamically tested to the performance requirements of ANSI/RESNA-WC 19 and ISO Standard 7176-19 and installed, used and maintained according to the manufacturer's instructions.
- If the original wheelchair seat was replaced, the new seat must be approved for wheelchair transit, installed and used as indicated in the manufacturer's instructions.
- The Wheelchair Tie-down and Occupant Restraint System (WTORS) should be compatible with the specific wheelchair, used as indicated in the manufacturer's instructions and should comply with the performance requirements of ISO Standard 10542.
- The Wheelchair must be forward facing during transport.
- If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

⚠ WARNING!

- The owner of this equipment is responsible for making sure that it has been set up and adjusted by an authorised dealer under the advice of a healthcare professional.
- The back and hardware may require periodic safety and function checks.
- Adjustments should be performed by your authorised dealer.
- Always use parts and/or accessories that have been recommended or approved by Sunrise Medical.

A. MAINTENANCE AND CLEANING

- Proper maintenance and cleaning will improve performance and extend the useful life of your equipment.
- Clean your equipment regularly. This will help you find loose or worn parts and make your equipment easier to use. You will need a mild detergent solution and cleaning rags.
- If discovered, have loose, worn, bent or damaged parts replaced before using the back.

B. CRITICAL SAFETY CHECKS **Fasteners:**

- Sunrise Medical recommends that all fasteners be checked for wear, such as loose bolts or broken components every 6 months.
- Loose fasteners should be re-tightened according to the installation instructions.
- Contact your authorised dealer immediately to replace any components.
- Do not continue to use the system after identifying loose or broken components.

C. EASY CLEANING (FIG.6.0)**1. Back cover**

- Remove the cover from back-shell by detaching the Velcro®.
- Unzip the zipper on the bottom of the cover.
- Remove the foam and turn the cover inside out.
- Zip the cover closed to avoid snagging.
- Machine wash in warm water (50°C).
- Drip and tumble dry in on Low Heat.

⚠ CAUTION!

- Do not dry clean covers or use industrial washers or dryers to clean covers.
- Do not steam autoclave.
- Do not bleach.
- Laundry care symbol definitions are shown.

2. Back shell

The back-shell should be wiped clean with soap and water on a regular basis to prevent build up of dirt and grime.

3. Positioning components

- Remove the components one at a time, noting the placement for re-assembly.
- Positioning components can be wiped clean with soap and water.
- DO NOT immerse positioning components underwater.

Fig.6.0**Laundry Care Symbols**

Machine wash in warm water (50° C)

Only non-chlorine bleach, when needed

Drip dry

Do not iron


Do not dry clean

D. WHEELCHAIR DIMENSIONS

1. The JAY Fit is designed to replace the wheelchair's sling upholstery. Prior to installation determine if the wheelchair has compatible push handles, by measuring the width between your push handles as seen in Fig.8.0.
2. Next check that the push handles are a compatible diameter. The JAY Fit hardware can be mounted on wheelchair push handles ranging from 19.0 mm, (3/4") to 32.0 mm, (1-1/8"). Measure your back cane diameter (Fig.7.0) and refer to Fig.9.0 to ensure your push handles are compatible.
3. If you cannot determine that the push handles are compatible please contact your authorised Sunrise Medical provider or Sunrise Medical customer service.

Fig.7.0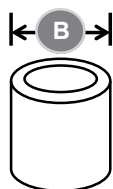

JAY Fit Width (all contours)	Push Handle Diameter (B)	
	Minimum Width	Maximum Width
254 mm - 457 mm (10" - 18")	19.0 mm (3/4")	28.5 mm (1-1/8")

Fig.8.0

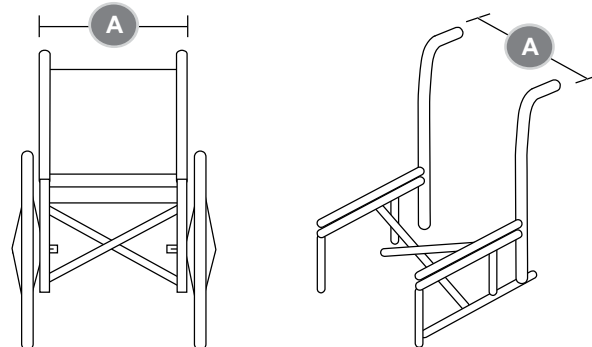
Fig.9.0

JAY Fit Width (all contours)	Push handle width measured from the outside edges of the tubes (A)	
	Minimum Width	Maximum Width
254 mm (10")	246 mm (9.7")	312 mm (12.3")
305 mm (12")	297 mm (11.7")	363 mm (14.3")
356 mm (14")	349 mm (13.7")	413 mm (16.3")
406 mm (16")	400 mm (15.7")	464 mm (18.3")
457 mm (18")	451 mm (17.7")	545 mm (20.3")

E. HYGIENE MEASURES WHEN BEING RE-USED

Prior to the wheelchair being re-used, it must be carefully prepared. All surfaces which come into contact with the user must be treated with a disinfection spray. To do this, you must use a disinfectant as authorised/recommended in your country, for rapid alcohol-based disinfection for medical products and medical devices, which must be disinfected quickly. Please be aware of the manufacturer's instructions for the disinfectant you are using. In general, a complete disinfection cannot be guaranteed on seams. We therefore recommend that you dispose of seat and back slings to avoid micro-bacterial contamination with active agents according to your local infection protection law.

F. CONDITIONS FOR SAFE STORAGE

- Products should be stored in a dry place: at 20 – 75% Relative Humidity.
- Protect from direct sunlight and dust for long periods.
- Away from direct heat and at a temperature between 5°C (40°F) to 40°C (104°F)
- Position and stack the packaged products according to the "this side up" arrows on the packaging.
- If a package is high and narrow and therefore potentially unstable, ensure it is well secured on the pallet or in the racking, to prevent it from falling down.
- Do not stack anything on top of the packaged products when the "do not stack" mark on the packaging is shown.
- Product should never be stored outside and subject to the elements.

A. FITTING THE JAY FIT BACK

⚠ WARNING!

- Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backward, potentially resulting in injury.
- Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

NOTE: The Jay Fit Back is designed for paediatric wheelchair clients. The back is very easy to adjust based on a child's changing needs. When installing or adjusting components, tighten all fasteners to 2.3 N-m (20 in.-lbs.).

B. ATTACH ONE LOWER RECEIVER ASSEMBLY TO EACH UPRIGHT POST OF THE WHEELCHAIR

1. Remove the black caps on the lower receiver assemblies (B) and release the bands by removing the band screws (C).
2. Position the lower receiver assemblies on the back posts in the approximate desired locations. HINT: These assemblies may be oriented in front or behind the back posts. The stud slot always faces inward. When mounted behind the back post, the assembly offers 0 cm to 3.7 cm (0-1.5") of forward adjustment. When the assembly is mounted in front of the back post, you have an adjustment range of 3.7 cm to 7.5 cm (1.5" - 3") forward.
3. Replace the band screw and tighten firmly. Replace the black cap.
4. To adjust the height of an assembly, loosen the adjustment screw until the receiver slides freely on the back posts. Tighten the adjustment screw when the proper height is determined. Both receivers should be set to the same height.

C. INSTALL THE JAY FIT BACK

1. Place the Jay Fit Back in the wheelchair so the bottom stud nuts (A), on the brackets attached to the back shell, slide into the lower receiver assemblies. If the width is not right, see next step "D" overleaf.
2. The top of the back should rest against the uprights. The top hooks on the upper portion of the back shell, should rest flush against the posts.

Fig.10.0


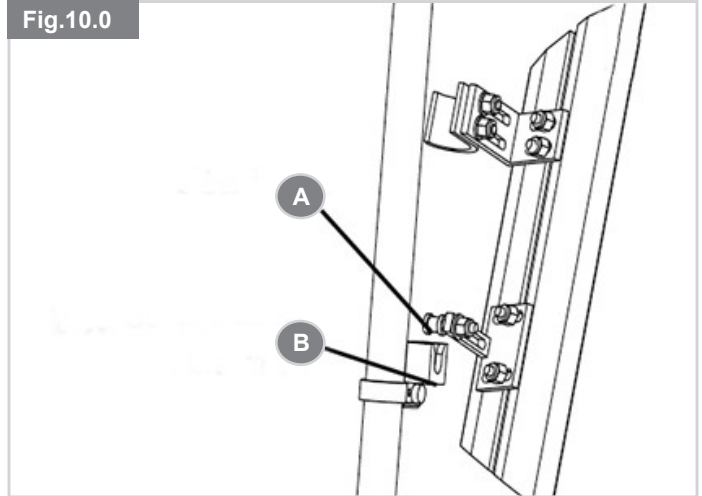
Fig.11.0

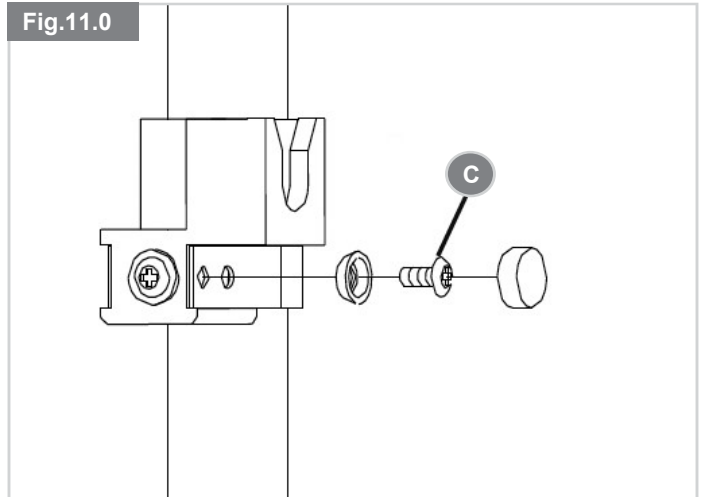- A** Stud Nut.
- B** Lower Receiver Assembly.
- C** Band Screw.

D. BOTTOM WIDTH ADJUSTMENT

Remove the Jay Fit Back from the wheelchair. Loosen the lower back bracket nuts and slide the brackets so that the stud nuts (E) fit into the lower receiver assemblies (B, previous page). Adjust the brackets evenly on both sides and lightly tighten the bracket nuts.

E. TOP WIDTH ADJUSTMENT

Remove the Jay Fit Back from the wheelchair. Loosen the upper back bracket nuts (D) and slide the brackets so that the upper back hooks match the wheelchair post width. Adjust the brackets evenly on both sides and tighten the bracket nuts. Replace the back in the wheelchair.

F. ASSESS AND ADJUST BACK HEIGHT

Position the user's hips to the back of the wheelchair. The top of the back should be located approximately at the top of the shoulder. To adjust the height of the back, loosen screw (F) and slide lower receiver assemblies up or down. Once proper height is determined, tighten brackets firmly. HINT: If you encounter interference with the lower back brackets, simply loosen the brackets at the point of attachment to the back, and move up or down in the track as needed.

G. ASSESS AND ADJUST SEAT DEPTH AND BACK ANGLE

1. To adjust seat depth, loosen the fasteners for the top hook and lower receiver stud nut on both sides (G). Slide the back on the bracket nuts to the desired depth.
2. To add recline, slide the top hooks forward or back to find the desired angle of recline. Recline may also be obtained by adjusting the placement of the stud nut on the lower bracket.
3. Adjust the top hooks and lower receiver stud nut evenly on both sides and tighten the fasteners.

Fig.12.0

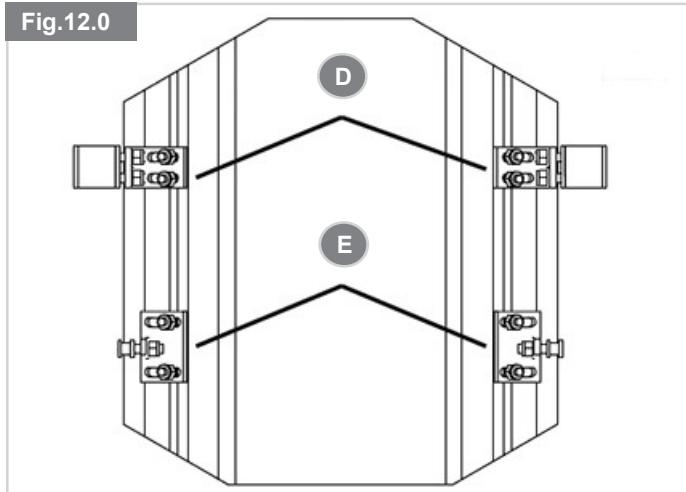
Fig.13.0


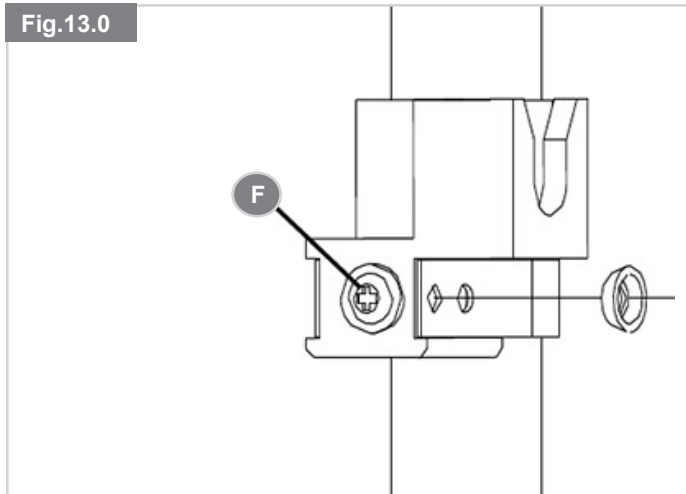
Fig.14.0

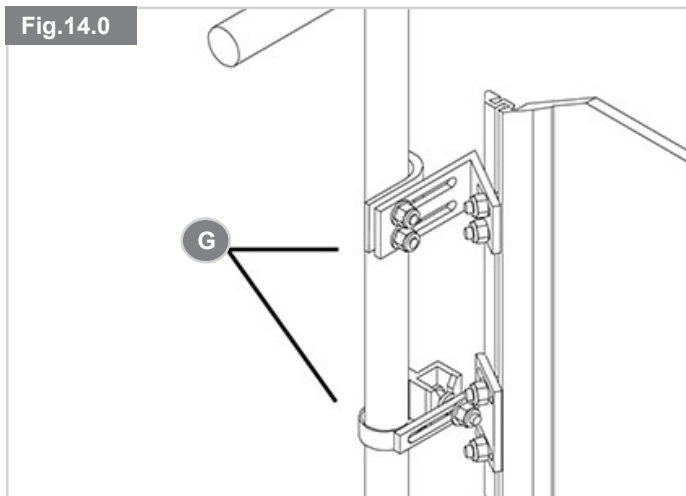- D** Upper Back Brackets.
- E** Lower Back Brackets.
- F** Adjustment Screw.
- G** Seat Depth and Angle Adjustment.

H. INSTALL THE TOP QUICK RELEASE QUARTER TURN BRACKET HARDWARE

Attach one top quick release quarter turn bracket (H) to each upright post of the wheelchair.

1. The top quick release quarter turn bracket is designed to fit on 1.6 cm, 1.9 cm, 2.2 cm, and 2.5 cm (5/8", 3/4", 7/8" and 1") diameter back post tubing. (The bracket comes assembled to attach to 2.2 cm or 2.5 cm tubing.) Check Fig.15.0 to determine which strap hole is appropriate for the installation. Wrap the strap around the wheelchair post. Screw (K) may have to be loosened to align the strap hole with the barrel nut located inside the bracket body. Place a washer on screw (J) and insert the screw through the appropriate hole in the strap.
2. Tighten screw (J) into the barrel nut through the side of the bracket body. Tighten the opposite screw if necessary.
3. Snap the screw caps over both screw heads.
4. To adjust the height of the bracket, loosen screw "A" until the bracket slides freely on the back post. Once the proper height of the back is determined, tighten the screw. All future adjustments of the back height are done with screw (K).
5. Always turn the quick-release quarter turn bracket to the "DOWN" position to secure the top hook in place.

NOTE: If you encounter interference with the top back brackets, simply loosen the brackets at the point of attachment to the back, and move up or down in the track as needed.

⚠ WARNING!

- The top quick-release quarter turn brackets must be in their downward position when the back is in use. Improper use may cause the back to unexpectedly detach from the wheelchair.
- Do not use the Jay Fit Back to push or lift the wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

Fig.15.0


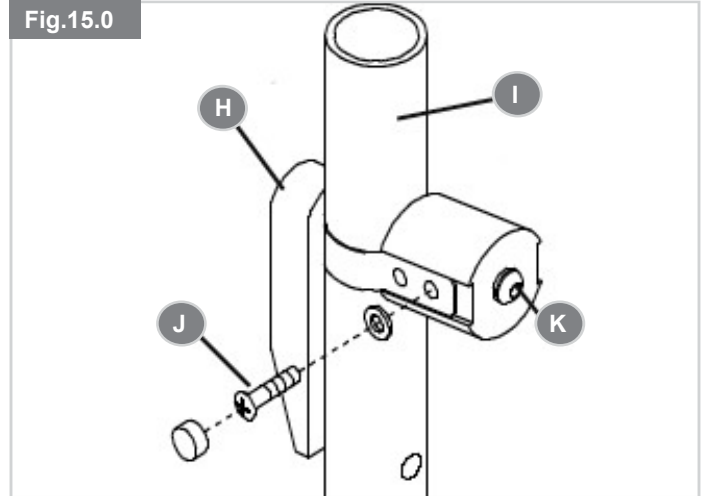
Fig.16.0

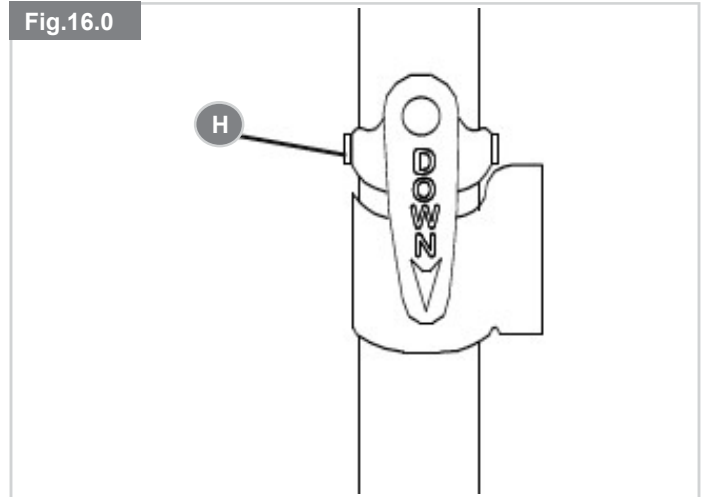- H** Quick Release Quarter Turn Bracket.
- I** Wheelchair Post.
- J** Fixing Screw.
- K** Adjustment Screw.

I. ADD OR ADJUST LATERAL/POSTERIOR PELVIC SUPPORT AS REQUIRED

To obtain proper placement, we recommend placing the individual in the wheelchair with the back adjusted to the desired height.

Have the person lean forward at the waist and position the lateral/posterior pelvic support so that the maximum thickness is immediately at the belt line.

Remove the foam insert from the back by unbuttoning the two snap buttons located at the bottom of the shell. Add or remove the lateral/posterior contour strips as needed for the client. Place the lateral pyramids on the left and right ends of the lateral/posterior contour strips for additional lumbar support.

The mounting track for the lateral/posterior contour strips may be raised or lowered to match the client's needs by loosening the mounting track hex nut, sliding in the track, and re-tightening where needed (L).

NOTE: The lateral/posterior pelvic supports may be difficult for clients to tolerate initially. Give the individual time to get accustomed to the support before giving up on it. In the long run, it may encourage proper posture and enhance sitting stability.

J. ASSESS AND FIT THE LATERAL THORACIC SUPPORTS

Assess client for proper positioning of the lateral thoracic supports. Each support is independent and may be adjusted for your client's clinical need.

1. To adjust the height, angle, or width of the lateral thoracic supports, remove the foam inserts, loosen the internally mounted bolts (M) and slide in the track to the desired height and width. The angle is obtained by tilting the bracket as needed. Tighten the bolts.
2. To adjust the depth of the lateral thoracic pads, remove the pad covers, loosen the attaching bolts, slide to the desired depth, re-tighten the bolts, and replace the pads.

⚠ WARNING!

A minimum of 25 mm (1") of clearance should exist between the top of the lateral support and the arm pit.

Fig.17.0


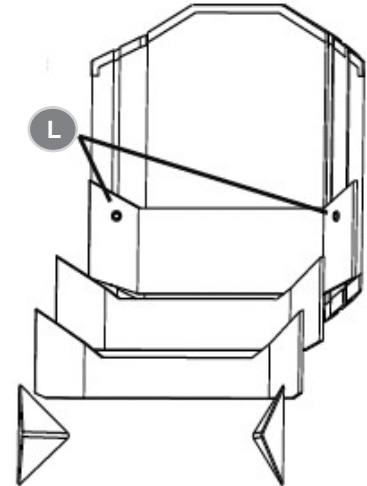
Fig.18.0

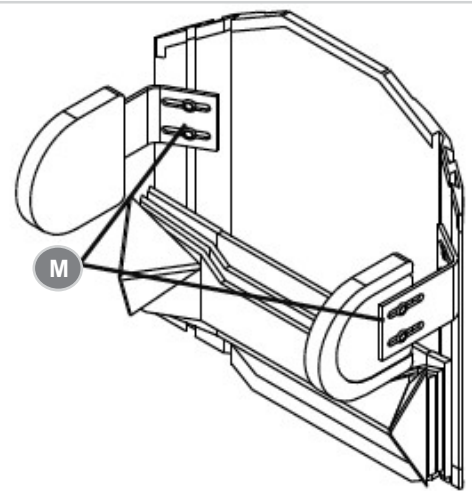K. MOUNTING AN OPTIONAL HEADREST

The Jay Fit Back has a universal headrest plate (N) mounted to it. Care should be taken to use the proper screw length when mounting.

L. SECURE THE FOAM AND COVER TO THE SHELL


To secure the back foam and cover on the Jay Fit Back, slide the top edge of the cover over the top lip of the shell. Press the back in place against the shell, pull the two button snaps under the bottom of the shell and snap to the buttons.

M. REMOVAL AND REPLACEMENT

Following installation and adjustment, the back can be easily removed from the wheelchair if necessary. Simply twist the quick release brackets (H - Fig.16.0), out of the way on both sides and lift the back up and out of the chair.

To replace, lower the stud nuts into the lower receiver assembly until it rests against the upright posts. Twist the quick release brackets down over the top hooks on the back.

The back will retain angle adjustment when removed and replaced on the wheelchair.

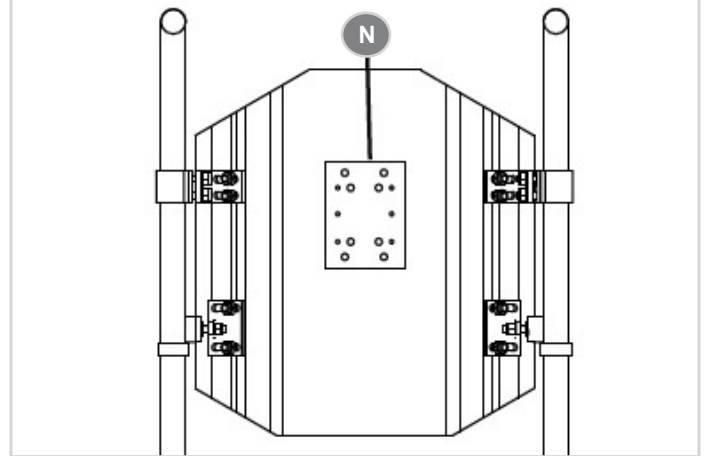Fig.19.0

N. HEIGHT ADJUSTABLE QUICK MOUNT ATTACHING SEAT HARDWARE

To Remove the Seat:

NOTE: Fig.20.0 viewed from underneath, Fig.21.0 and Fig.22.0 are viewed from the side.

1. With the chair unoccupied, reach under the front of the seat and pull the quick release cable (P) forward.
2. While holding the release cable forward, lift the front of the seat from the latch mechanism assembly (O) and slide the rear brackets (Q) forward from the rear receiver clamp assemblies (R) on the chair.

To Attach the Seat:

1. Holding the seat as shown in Fig.21.0, slide the rear brackets (Q) into the rear receiver clamp assemblies (R).
2. Centre the latch mechanism assemblies onto the front receiver studs (S) and press down until locked.
3. Insert the safety lock pin (T) into the latch mechanism assembly.

NOTE: For a wheelchair with the Sunrise Transit Option, your system will have two lock pins. Insert the safety lock pins into the latch mechanism assemblies (S) on both sides of the seat during transit.

WARNING!

Risk of injuries and how to avoid them:

Improper attachment of the seat may cause the seat to unintentionally disconnect during use, potentially causing the rider to fall.

To avoid the risk:

- It is vital to follow all instructions when re-attaching the seat.
- Before seating the rider, ensure that the seat is locked in place.
- Return to your supplier for adjustment if the latch mechanism assemblies do not centre and lock on the front receiver studs with ease.

Fig.20.0

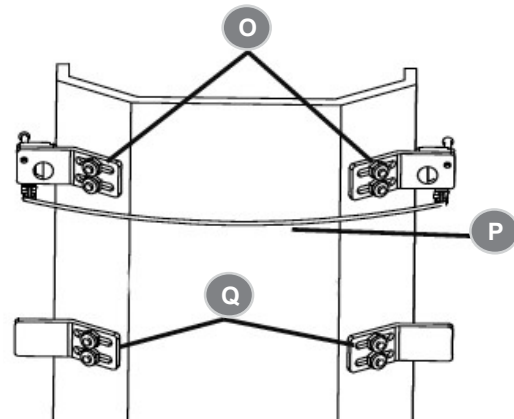
Fig.21.0


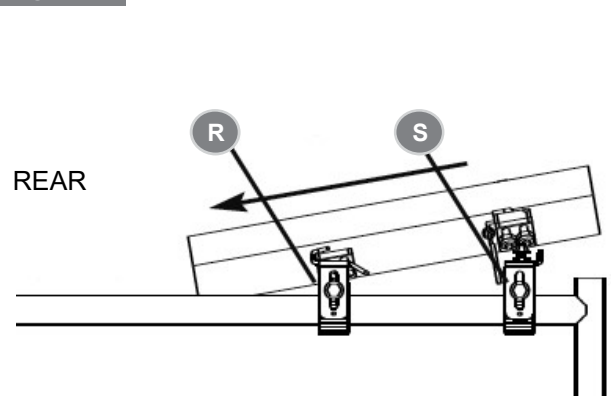
Fig.22.0

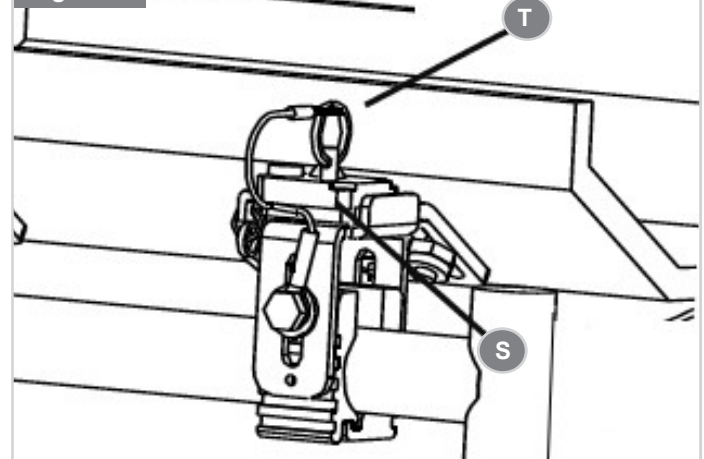O. ADJUSTABLE QUICK MOUNT ATTACHING BACK HARDWARE

To Remove the Back:

1. With the chair unoccupied, release the SnaPtite hinge (V).
2. For a wheelchair with a Transit Option, you must release the safety retainer strap (W) first.
3. Tilt the top of the back forward and lift from the lower back receivers (Y).

To Attach the Back:

1. Holding the back, slide the lower stud bracket nuts (X) into the lower back receivers (Y).
2. Press the SnaPtite attaching hardware around the SnaPtite receiver (U) and snap the hinge (V) shut.
3. For a wheelchair with the Sunrise Transit Option, you **MUST** re-attach the safety retainer strap (W) by pulling it around the SnaPtite hinge (V).

⚠ WARNING!

Risk of injuries and how to avoid them:

- Failure to lock the back in place on the chair back post, may cause the back to disengage from the chair frame, potentially causing the rider to fall.

To avoid the risk:

- It is vital to follow all instructions when re-attaching the back.
- Before seating the rider, ensure that the back is locked in place.
- Return to your authorised supplier for adjustment if the SnaPtite hinge does not close with ease around the receiver, or if the safety retainer strap does not fit around the hinge.

Fig.23.0

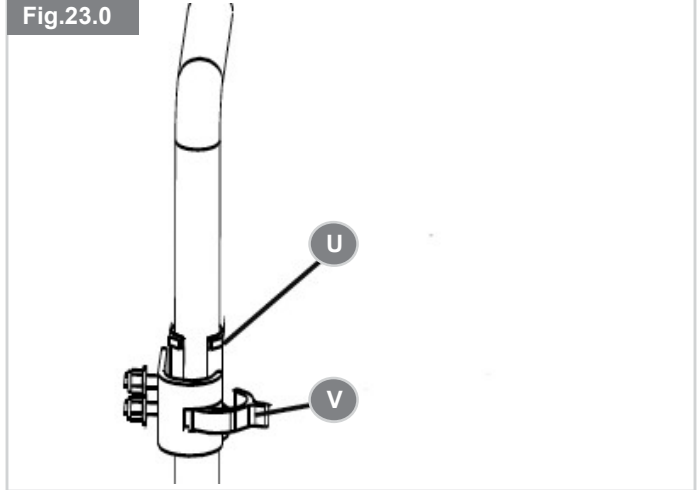
Fig.24.0


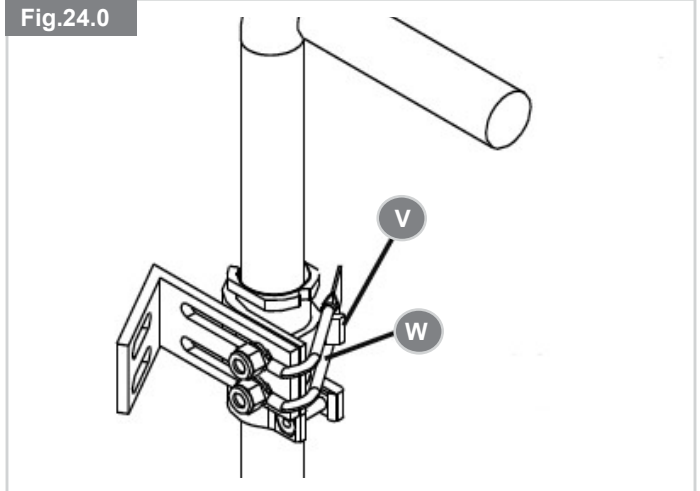
Fig.25.0

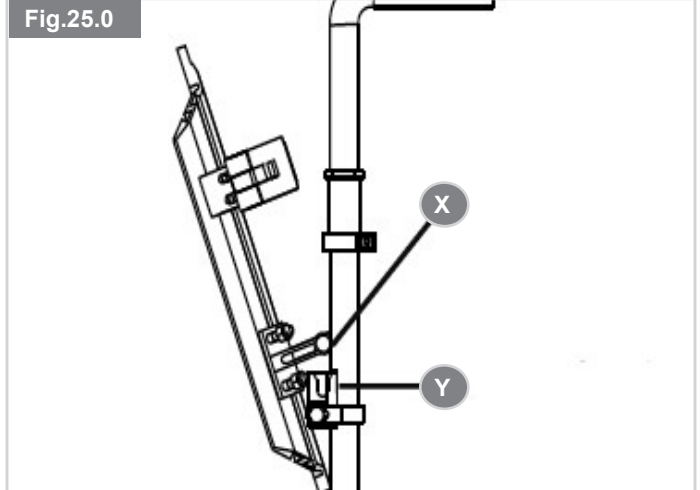P. QUICK MOUNT SEAT HARDWARE CHECK LIST

- Check that all the fixings are in place and correctly fitted.
- Check that the Quick Release Quarter Turn is engaged downwards in the locked position.
- Confirm that the Stud Post is seated in the receiver.
- Check that the Rear Seat Hook Mounts are firmly located under the retaining hardware.
- Check that the Front Seat Hook Mounts are firmly located against the seat stop.

Fig.26.0

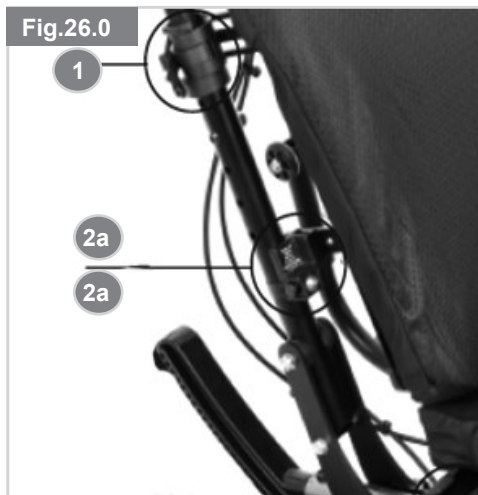
Fig.28.0

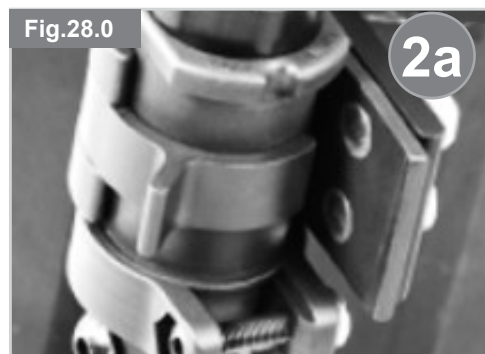
Fig.29.0

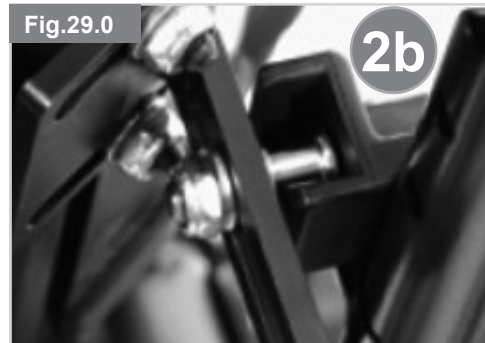
Fig.27.0

ACCESSORIES

A. SPINE-ALIGN POSITIONING COMPONENTS

Spine Align Positioning Components are designed to provide intimately shaped support with-in an “off-the-shelf” back. Depending on your market, Spine Align components may be included with your back.

1. Spine Align Positioning Component installation

- Have the user remove pressure from the Back by leaning or sliding partially forward if possible.
- Remove the top cover pocket and lift the cover forward to reveal the inside of the JAY® Fit Back.
- Place the positioning component(s) on the Velcro inside of the shell and reattach the top cover pocket.
- Have the user test the current Spine Align setup for comfort and support and repeat steps a-c until adjustment is complete.
- Multiple configurations and combinations of Spine Align components can be used. If additional Spine Align shapes or sizes are required, please contact your Sunrise Medical authorised dealer.
- Mounting an optional head rest. Head rests can be mounted to the JAY Fit Back, using the head rest mounting plate and head rest mount.
- Securing the foam and cover to the shell.
 - To secure the back foam and cover on the JAY® Fit System back, slide the top edge of the cover over the top lip of the shell.
 - Press the back in place against the shell.
 - Pull the two button snaps under the bottom of the shell and snap into the buttons.

NOTE: If you require further information on the JAY® Fit Spine Align system, please review our animation at www.SunriseMedical.com.

Positioning Components	
Large Lumbar	
Small Lumbar	
Wedge	
Curved Shim	
Flat Shim	
Posterior/Lateral Pelvic	
Lateral Shim	
Large Bean Bag	
Small Bean Bag	

24 MONTH LIMITED WARRANTY

- Each JAY® Fit Back is carefully inspected and tested to provide peak performance. Every JAY® Fit Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the back's removable cover.
- Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.
- For goods provided by Sunrise Medical Pty Ltd in Australia, our goods come with a guarantee by Sunrise that cannot be excluded under the Australian Consumer Law. You are entitled to replacement or refund for a major failure & for compensation for any other reasonable foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality & the failure does not amount to a major failure. The benefits to you given by this warranty are in addition to your other rights & remedies under a law in relation to the goods to which the warranty relates.
- If you have any questions: Contact Sunrise Medical Customer Service.

Type:	Product Name/ SKU Number	 XXXX-XX-XX	Date of Manufacture
Date:	LOT: defined by Julian Date	Part Code	Assembly Part Number
 XXX mm	Seat width	MD	This symbol means Medical Device
 XXX mm	Seat depth		Manufacturer's address
 XXX kg	Max user weight	EC REP	European Authorised Representative
CE	CE Mark	UK RP	UK Responsible Person
UK CA	UKCA Mark	CH REP	Swiss Representative's address
	Consult instructions for use		Importer's address

SAMPLE

JAY

Part Code:

Date: 

cm cm kg

 Sunrise Medical (US) LLC
 2842 Business Park Avenue
 Fresno, California, 93727 USA
 Sunrise Medical GmbH
 Kahlbachring 2-4
 69254 Malsch / HD
 Germany
 Sunrise Medical Ltd.
 Thorns Road, Brierley Hill
 West Midlands DY5 2LD
 United Kingdom
 Assembled in Mexico

EC REP **UK RP**

106132 Rev. J 

The management system of SUNRISE MEDICAL is certified to EN ISO 13485 and ISO 14001.

We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R & D to production. This product meets the requirements in accordance with EU and UK regulations. Options or accessories shown are available at extra cost.

The varieties of backrest fitting variants, as well as the different positioning components to deliver postural support, pressure distribution, increase seating tolerance are benefits according to each user requirement and indication, mean that it can be used by a range of users with different positioning needs and indications as follows:

- Spinal Cord Injury - Amputation - Joint contracture
- MS - Tetraplegia - Cerebral Palsy
- Neurological disease pattern (hemiplegia, M. Parkinson)
- Muscle Dystrophy - Extremity defect deformity

No contraindications are known or available.

NOTE: General user advice. Not following these instructions may result in physical injury, damage to the product or damage to the environment!

Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

As the manufacturer, SUNRISE MEDICAL, declares that the product conforms to the UK Medical Devices Regulation 2002 No. 618.

B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used. Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.Sunrisemedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, section 14.1 of the Medical Device Regulation 2017/745. Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk.

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S