


Seat cushion fixation set
Festesett for setepute
Sittdynans fästanordning

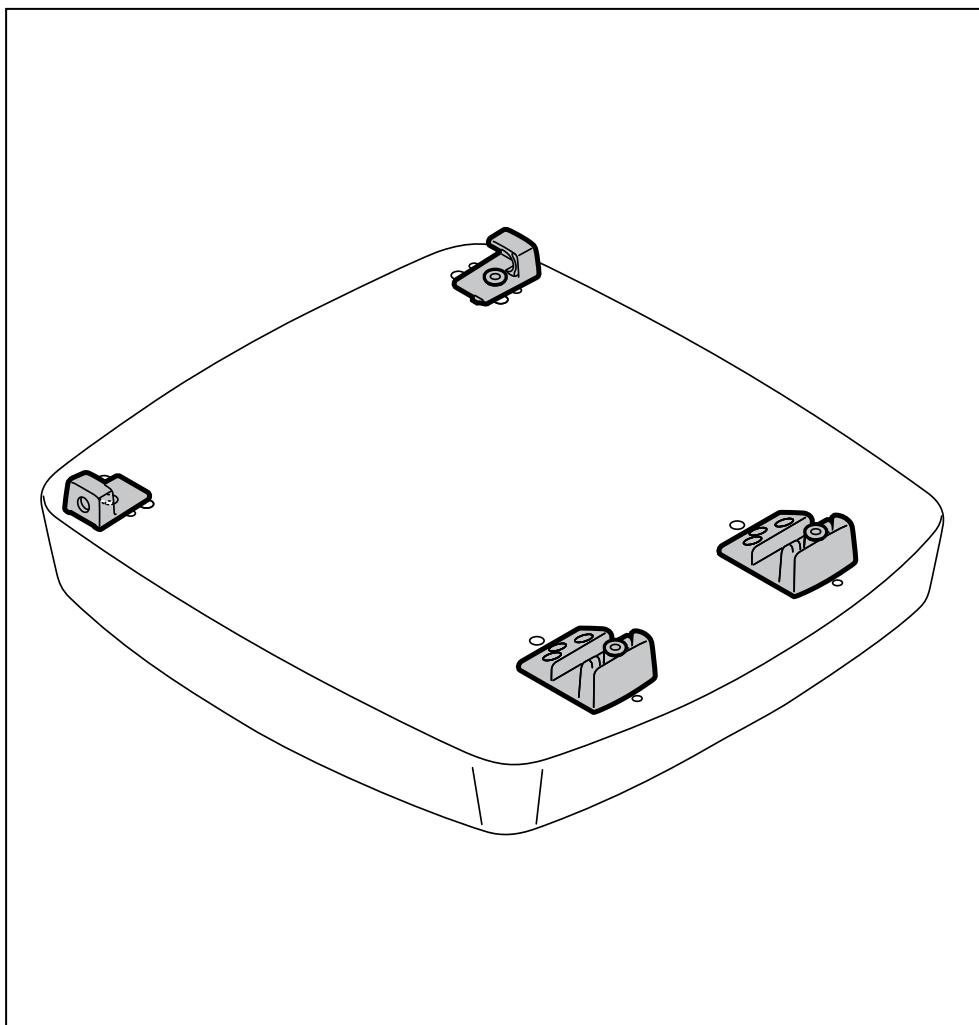

Seddeo Lite / Pro

Seddeo Pro+

Seat cushion fixation set

ASSEMBLY INSTRUCTIONS
MONTERINGSANVISNING
MONTERINGSANVISNING

EN




NO

SV

SEDEO

9010198C

1

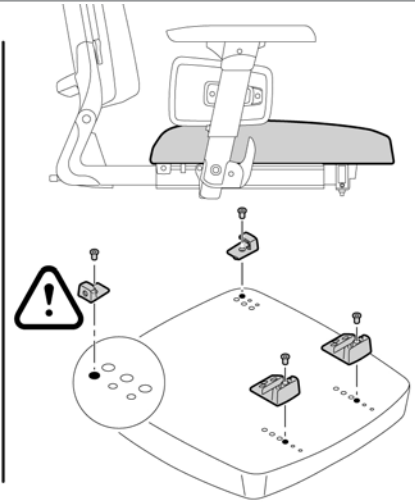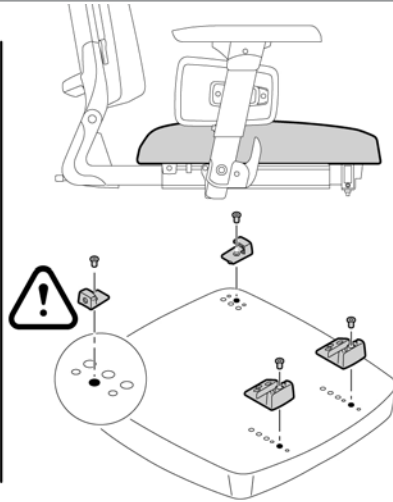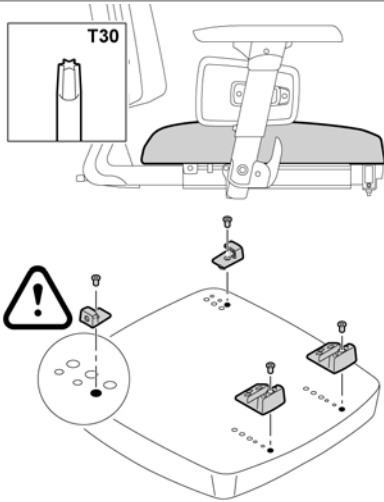

110648-06-020

Adjusting or mounting the seat cushion fixation set

Preparation

Note! Switch off the wheelchair via remote control.

Instructions

Note! Suitable work area required!

Follow the illustrated instructions for adjusting or mounting the seat cushion fixation.


Note! Ex-factory the fixation brackets on the cushion or seat are always mounted in the middle position! In case you want to reposition the seat cushion, place the fixation brackets at the front and backside always in the same holes! The seat board Sedeo SD40 has only two positions. Ex-factory the fixation brackets on the seat board are mounted in the back position.

Relevant article numbers

1010953 Seat cushion fixation set

1012267 Interface board Sedeo Original

Tools used

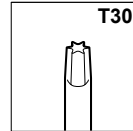

Justere eller montere festesett for setepute

Forberedelser

Merk! Slå av rullestolen via styreboksen

Anvisninger

Merk! Bruk et egnet arbeidsområde!

Følg de illustrerte instruksjonene for å justere eller montere seteputefestet.


Merk! Fra fabrikk er festebrakettene på puten eller setet alltid monter i midtre posisjon! Hvis du vil flytte på seteputen, plasserer du festebrakettene på for- og baksiden alltid i de samme hullene! Seteplate Sedeo SD40 har bare to posisjoner. Fra fabrikk er festebrakettene på seteplaten montert i bakre posisjon.

Relevante artikkelnumre

1010953 Festesett for setepute

1012267 Grensesnittplate Sedeo Original

Verktøy som brukes

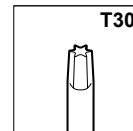

Justering eller montering av sittdynans fästanordning

Förberedelser

Obs! Stäng av manöverboxens av- och påknapp

Anvisningar

Obs! Lämpligt arbetsområde krävs!

Följ de illustrerade anvisningarna för justering eller montering av sittdynans fästanordning.


Obs! I fabriksskick är fästkonsolerna på dynan eller sätet alltid monterade i mittläget! Om sittdynans placering ska ändras måste de främre och bakre fästkonsolerna alltid placeras i samma håll! Sättesbrädan på Sedeo SD40 har endast två lägen. I fabriksskick är fästkonsolerna på sättesbrädan alltid monterade i det bakre läget.

Relevanta artikelnummer

1010953 Seat cushion mounting set

1012267 Interface board Sedeo Original

Verktyg som krävs

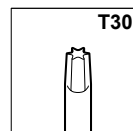

AU
Sunrise Medical Pty. Ltd.
6 Healey Circuit, Huntingwood
NSW 2148
Australia
T: +61 (2) 9678 6600
E: enquiries@sunrisemedical.com.au
www.SunriseMedical.com.au

BE
Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands
T: +32 (0)78 150 159
E: customerservice@sunrisemedical.be
www.SunriseMedical.be

CH
Sunrise Medical AG
Lückhalde 14
3074 Muri/Bern
Schweiz/Suisse/Svizzera
T: +41 (0)31 958 3838
E: info@sunrisemedical.ch
www.SunriseMedical.ch

DE / AT
Sunrise Medical GmbH & Co. KG
Kahlbachring 2-4
69254 Malsch/Heidelberg
Deutschland
T: +49 (0) 7253/980-0
E: kundenservice@sunrisemedical.de
www.SunriseMedical.de

DK
Sunrise Medical ApS
Park Allé 289 G
DK-2605 Brøndby
T: +45 70 22 43 49
Email: info@sunrisemedical.dk
www.SunriseMedical.dk

ES / PT
Sunrise Medical S.L.
Polígono Bakiola, 41
48498 Arrankudiaga – Vizcaya
España
T: +34 (0) 902142434
E: marketing@sunrisemedical.es
www.SunriseMedical.es

FR
Sunrise Medical S.A.S
ZAC de la Vrillonnerie
17 Rue Mickaël Faraday
37170 Chambray-Lès-Tours
T: + 33 (0) 2 47 55 44 00
E: info@sunrisemedical.fr
www.SunriseMedical.fr

IT
Sunrise Medical S.r.l.
Via Riva, 20 – Montale
29122 Piacenza
Italia
T: +39 0523 573111
E: info@sunrisemedical.it
www.SunriseMedical.it



NL
Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands
T: +31 (0)492 593 888
E: customerservice@sunrisemedical.nl
www.SunriseMedical.nl
www.SunriseMedical.eu (International)

NO
Sunrise Medical AS
Delitoppen 3
1540 Vestby
Norway
T: +47 66963800
E: post@sunrisemedical.no
www.SunriseMedical.no

PL
Sunrise Medical Poland Sp. z o.o.
ul. Elektronowa 6,
94-103 Łódź
Polska
T: + 48 42 275 83 38
E: pl@sunrisemedical.de
www.Sunrise-Medical.pl

SE
Sunrise Medical AB
Box 9232
400 95 Göteborg
Sweden
T: +46 (0)31 748 37 00
E: post@sunrisemedical.se
www.SunriseMedical.se

UK
Sunrise Medical
Thorns Road
Brierley Hill
West Midlands
DY5 2LD
United Kingdom
T: +44 (0) 845 605 66 88
E: enquiries@sunmed.co.uk
www.SunriseMedical.co.uk

Dealer / distributor:

Serial number:



MD_Sedeo_seat_fixation_set_EU_EN_NO_SE_RevC_2016_11_25

