


Backrest cushion fixation set
Festesett for ryggpute
Ryggdynans fästanordning

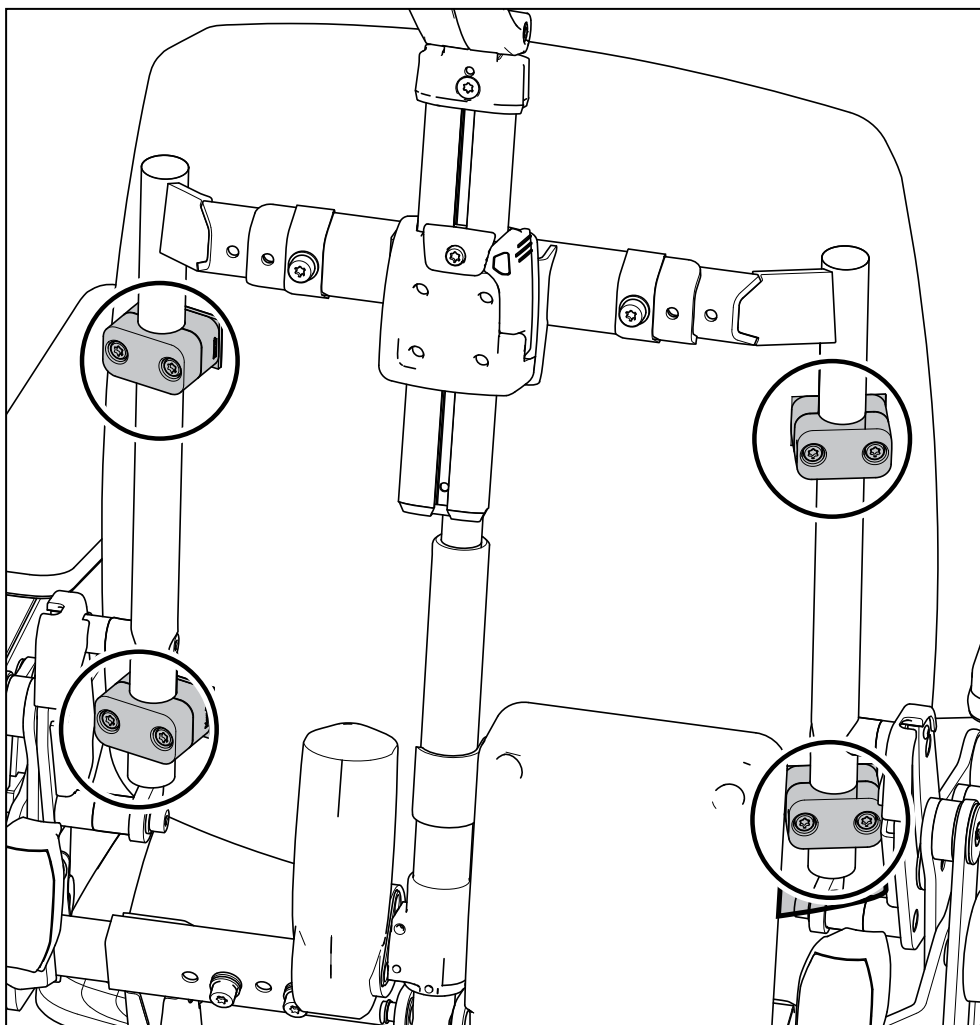

Sedeo Pro+

Backrest cushion fixation set


ASSEMBLY INSTRUCTIONS
MONTERINGSANVISNING
MONTERINGSANVISNING

EN
NO
SV

SEDEO

9010200C

1

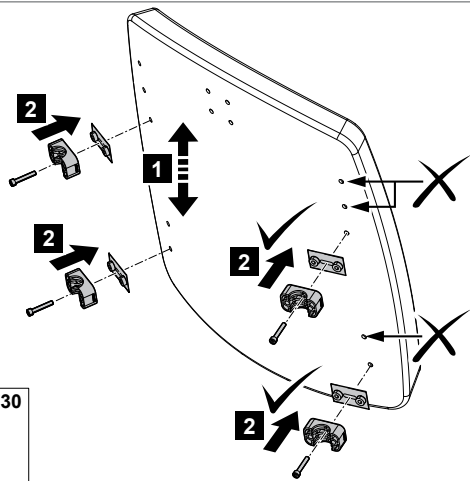

T30

Mounting the backrest cushion fixation set

Preparation

Note! Switch off the wheelchair via remote control.

Instructions


Note! Suitable work area required!

- **Note!** Define by feeling the lowest hole at the top of the frame and the lowest hole at the bottom of the frame.
- Rebuild afterwards the wheelchair in reversed order.

Relevant article numbers

1016821 Backrest mounting set

Tools used

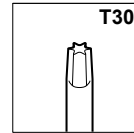

Montere festesett for ryggpute

Forberedelser

Merk! Slå av rullestolen via styreboksen.

Anvisninger


Merk! Bruk et egnet arbeidsområde!

- **Merk!** Finn, ved å føle deg fram, det laveste hullet på toppen av ramma og det laveste hullet ved bunnen av ramma.
- Sett sammen rullestolen igjen i motsatt rekkefølge.

Relevante artikkelnumre

1016821 Monteringssett for rygg

Verktøy som brukes

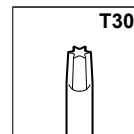

Montering av ryggdynans fästanordning

Förberedelser

Obs! Stäng av manöverboxens av- och påknapp

Anvisningar


Obs! Lämpligt arbetsområde krävs!

- **Obs!** Avgränsa genom att känna efter var det nedersta hålet ovanpå ramen och det nedersta hålet under ramen sitter.
- Montera sedan ihop rullstolen i omvänd ordning.

Relevanta artikelnummer

1016821 Backrest mounting set

Verktyg som krävs

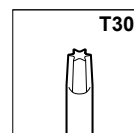

AU
Sunrise Medical Pty. Ltd.
6 Healey Circuit, Huntingwood
NSW 2148
Australia
T: +61 (2) 9678 6600
E: enquiries@sunrisemedical.com.au
www.SunriseMedical.com.au

BE
Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands
T: +32 (0)78 150 159
E: customerservice@sunrisemedical.be
www.SunriseMedical.be

CH
Sunrise Medical AG
Lückhalde 14
3074 Muri/Bern
Schweiz/Suisse/Svizzera
T: +41 (0)31 958 3838
E: info@sunrisemedical.ch
www.SunriseMedical.ch

DE / AT
Sunrise Medical GmbH & Co. KG
Kahlbachring 2-4
69254 Malsch/Heidelberg
Deutschland
T: +49 (0) 7253/980-0
E: kundenservice@sunrisemedical.de
www.SunriseMedical.de

DK
Sunrise Medical ApS
Park Allé 289 G
DK-2605 Brøndby
T: +45 70 22 43 49
Email: info@sunrisemedical.dk
www.sunrisemobility.dk

ES / PT
Sunrise Medical S.L.
Polígono Bakiola, 41
48498 Arrankudiaga – Vizcaya
España
T: +34 (0) 902142434
E: marketing@sunrisemedical.es
www.SunriseMedical.es

FR
Sunrise Medical S.A.S
ZAC de la Vrillonnerie
17 Rue Mickaël Faraday
37170 Chambray-Lès-Tours
T: + 33 (0) 2 47 55 44 00
E: info@sunrisemedical.fr
www.SunriseMedical.fr

IT
Sunrise Medical S.r.l.
Via Riva, 20 – Montale
29122 Piacenza
Italia
T: +39 0523 573111
E: info@sunrisemedical.it
www.SunriseMedical.it



NL
Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands
T: +31 (0)492 593 888
E: customerservice@sunrisemedical.nl
www.SunriseMedical.nl
www.SunriseMedical.eu (International)

NO
Sunrise Medical AS
Delitoppen 3
1540 Vestby
Norway
T: +47 66963800
E: post@sunrisemedical.no
www.SunriseMedical.no

PL
Sunrise Medical Poland Sp. z o.o.
ul. Elektronowa 6,
94-103 Łódź
Polska
T: + 48 42 275 83 38
E: pl@sunrisemedical.de
www.Sunrise-Medical.pl

SE
Sunrise Medical AB
Box 9232
400 95 Göteborg
Sweden
T: +46 (0)31 748 37 00
E: post@sunrisemedical.se
www.SunriseMedical.se

UK
Sunrise Medical
Thorns Road
Brierley Hill
West Midlands
DY5 2LD
United Kingdom
T: +44 (0) 845 605 66 88
E: enquiries@sunmed.co.uk
www.SunriseMedical.co.uk

Dealer / distributor:

Serial number:



MD_Sedeo_backrest_cushion_fixation_EU_EN_NO_SE_RevC_2016_11_25

