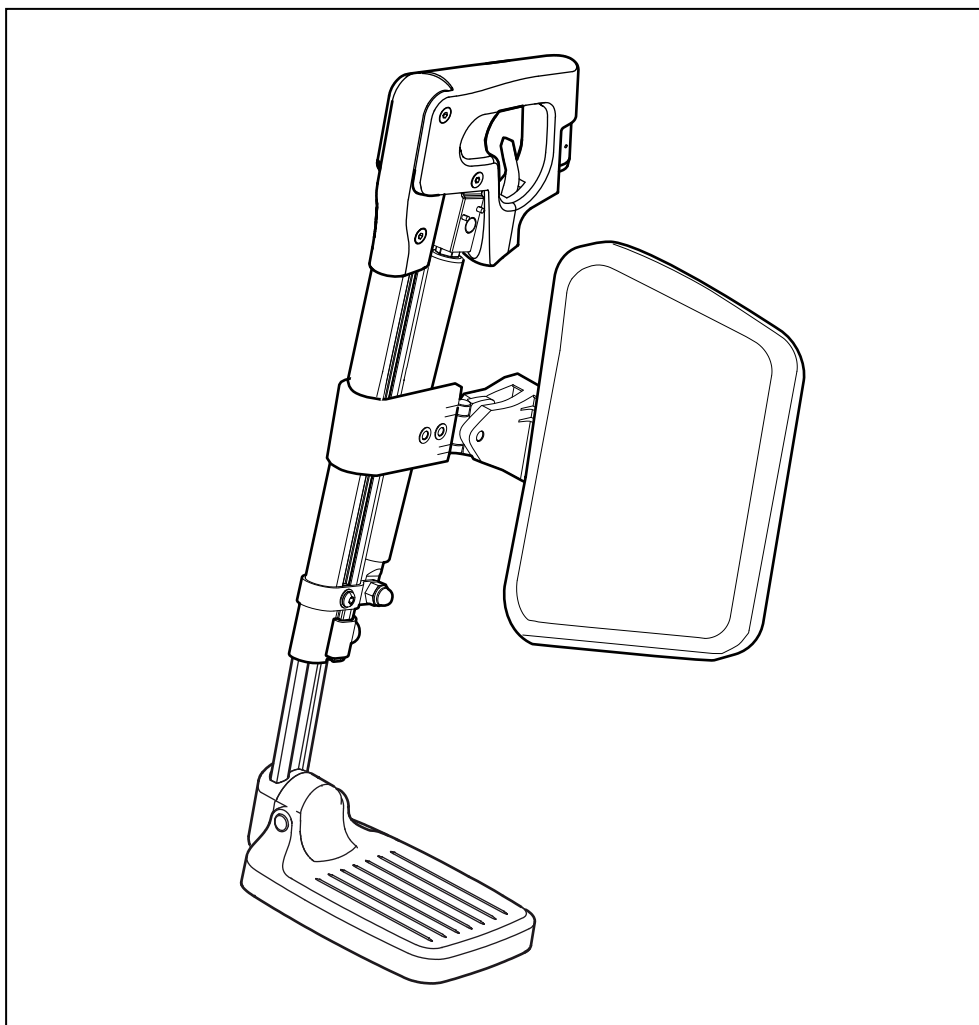
Manually elevating legrest
Manuelt vinkelstillbar benstøte
Manuellt höjbart benstöd

Sedeo Lite / Pro

Sedeo Pro+

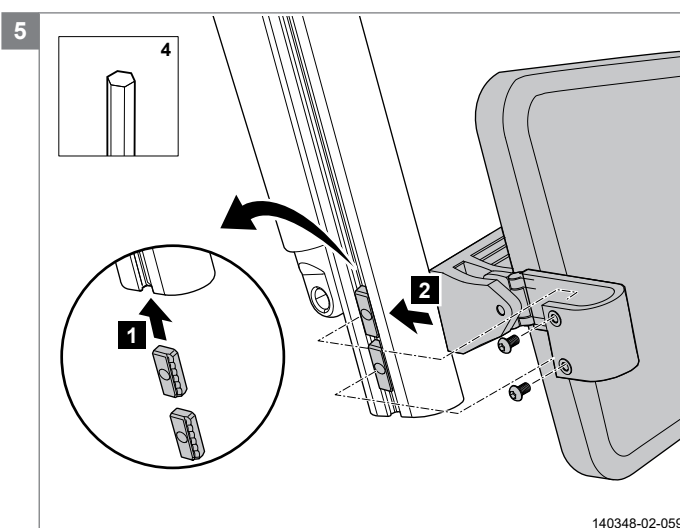
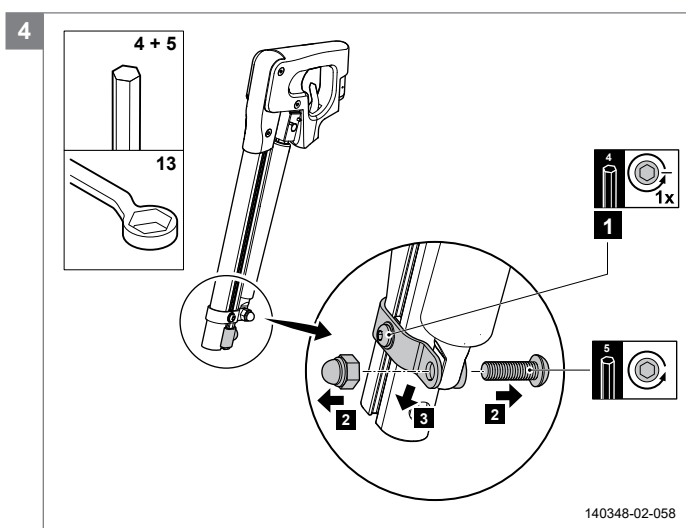
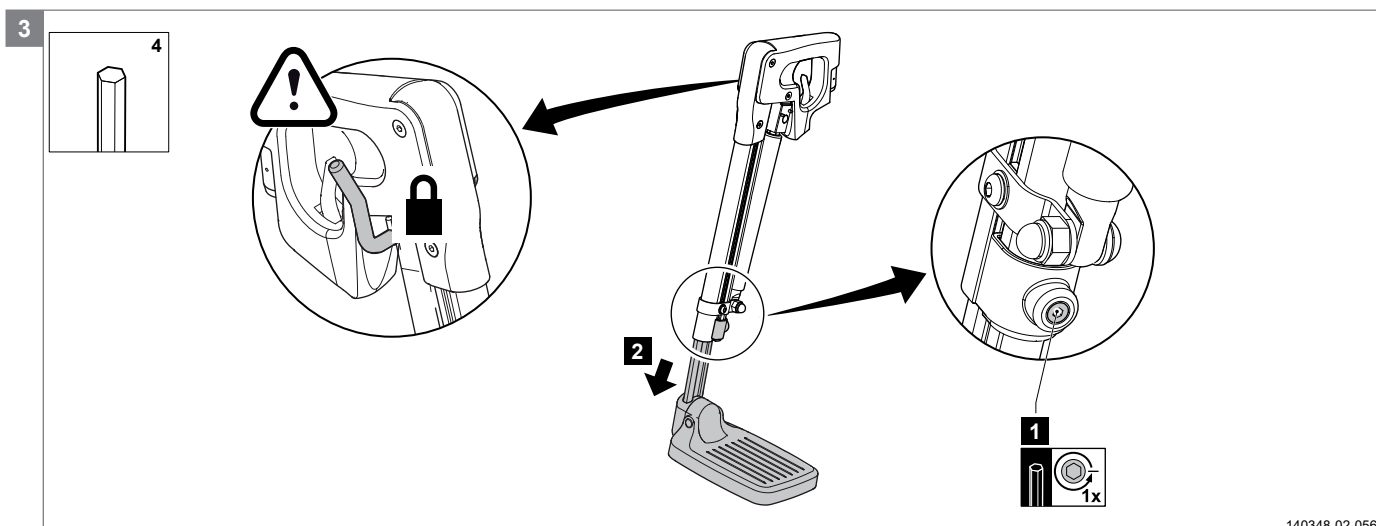
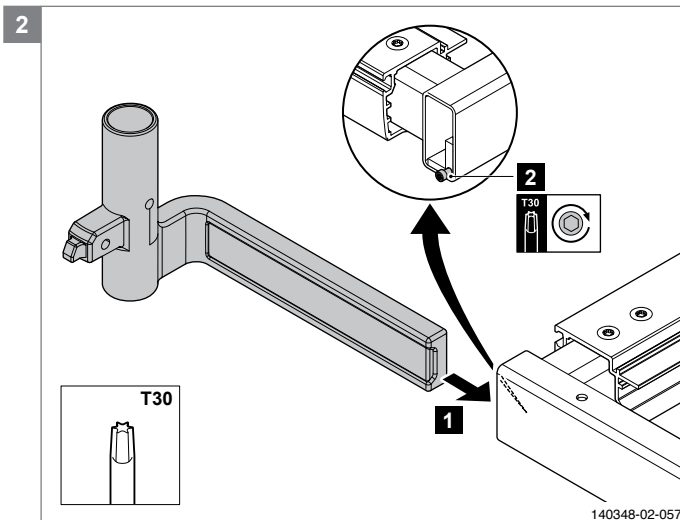
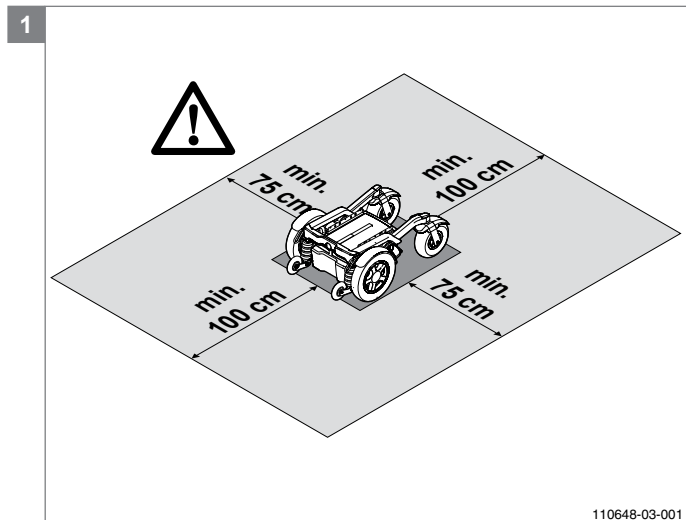
Manually elevating legrest

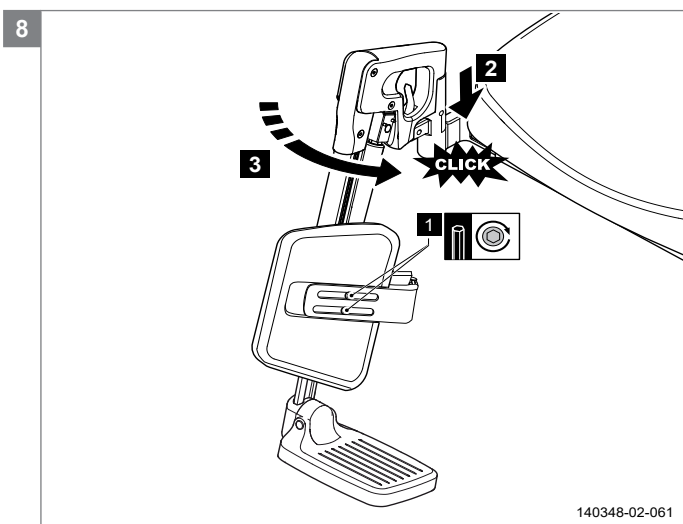
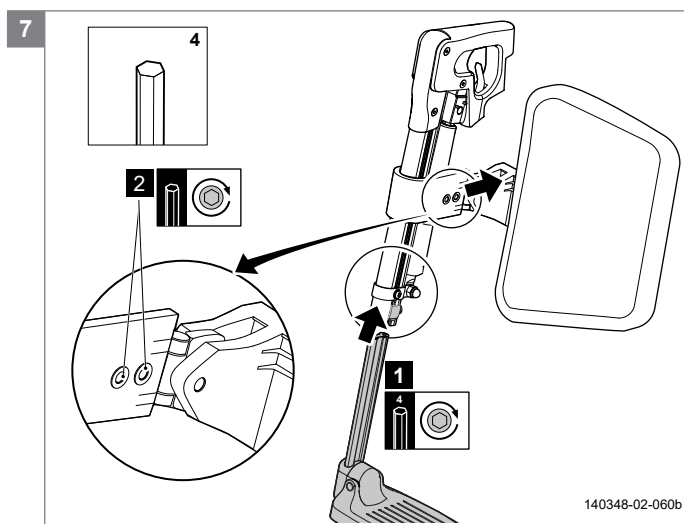
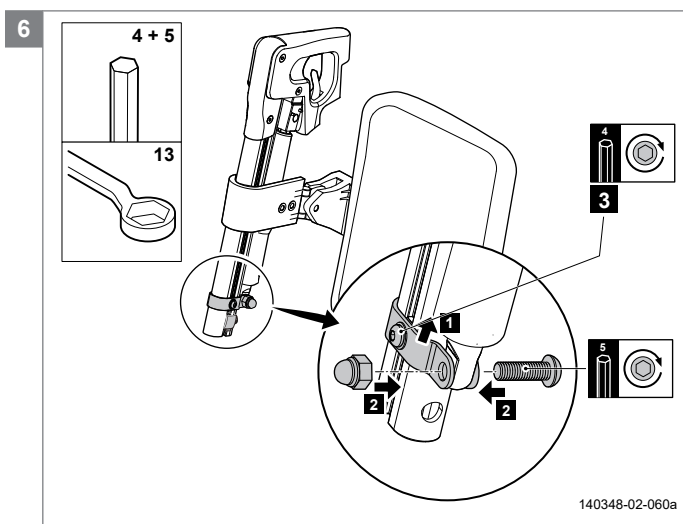
ASSEMBLY INSTRUCTIONS
MONTERINGSANVISNING
MONTERINGSANVISNING

EN
NO
SV

SEDEO

9010204C

Mounting the manually elevating legrest

Preparation

Note! Switch off the wheelchair via remote control.

Instructions

Note! Suitable work area required!

- Mount the legrest mounting bracket to the seat support frame.
- Secure the legrest in its upper position.
- Release the footplate fixing screw and lower the footplate to remove it from the legrest.
- Unscrew the lower clamp of the damper unit.
- Position the clamp nuts in the legrest profile and attach the calf plate to the clamp nut.
- Adjust the calf plate height to the desired position, tighten the clamp nuts.
- Remount the lower clamp of the damper unit.
- Reposition the footplate and tighten the fixing screw. Adjust the calf plate depth to the desired position.
- Adjust the calf plate inwards/outwards to the desired position. Lower the legrest into the tube of the mounting bracket, holding the footplate outwards. Swing the legrest inwards to lock it in position.

Note! Make sure the locking pin falls in place (with a 'click') and secures the legrest from becoming detached.

Note! If locking fails: Adjust the screw from the locking mechanism.

Relevant article numbers

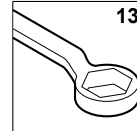
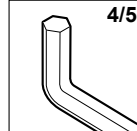
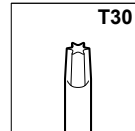
1018305 Manually elevating legrest set with calfplates SW 42-46

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Sunrise Medical AB