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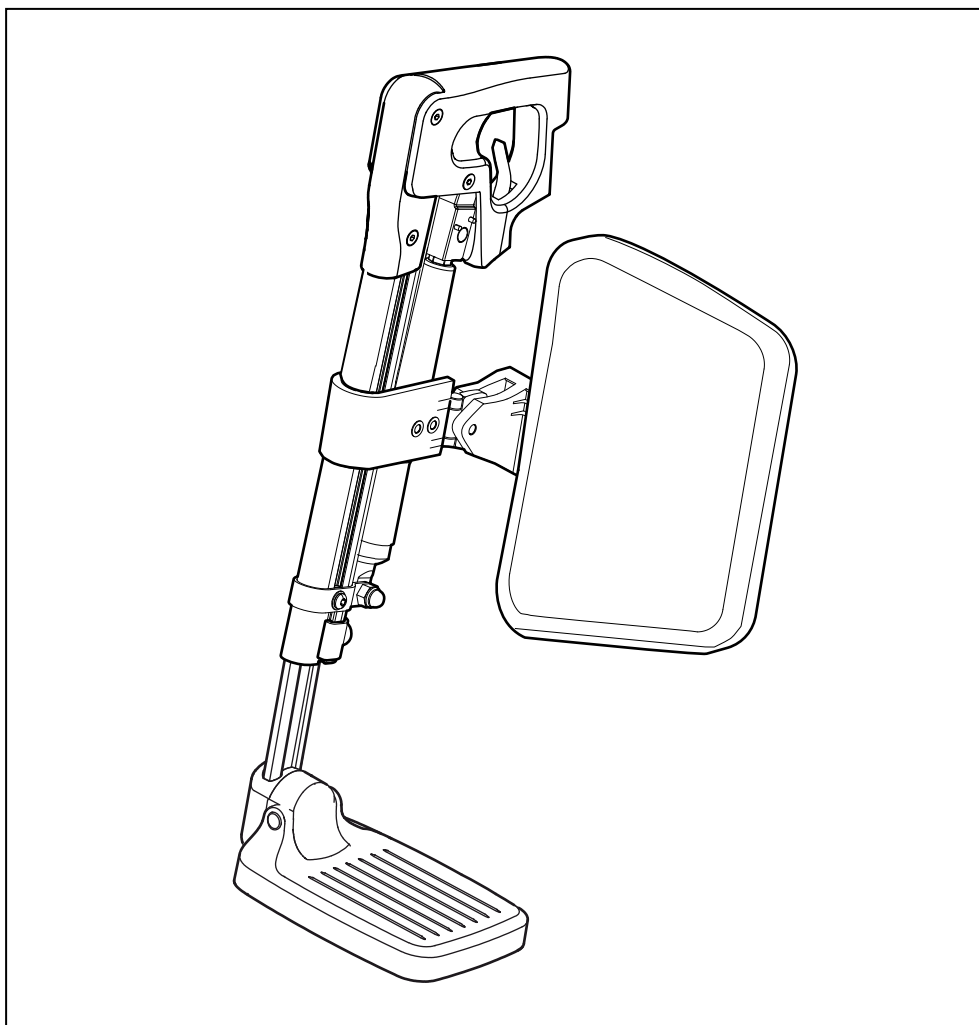
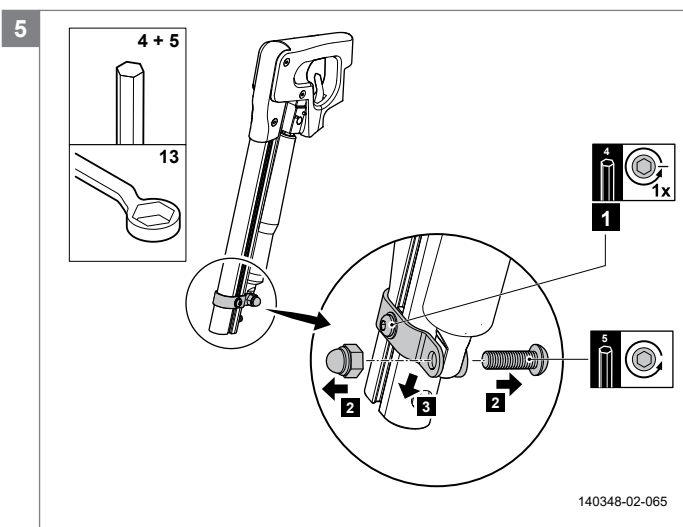
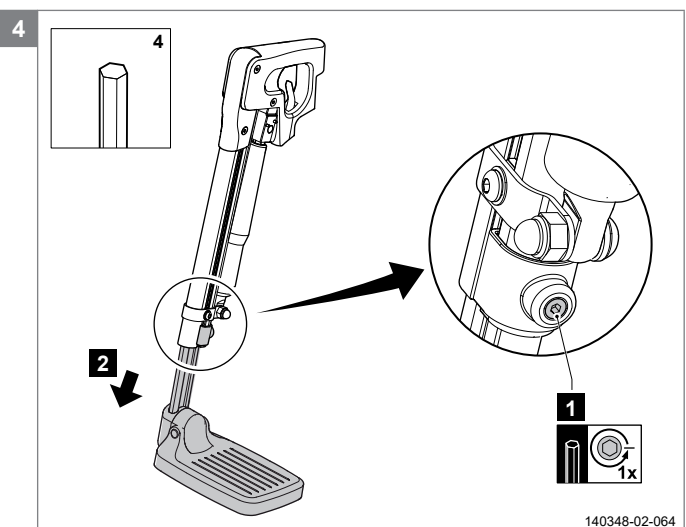
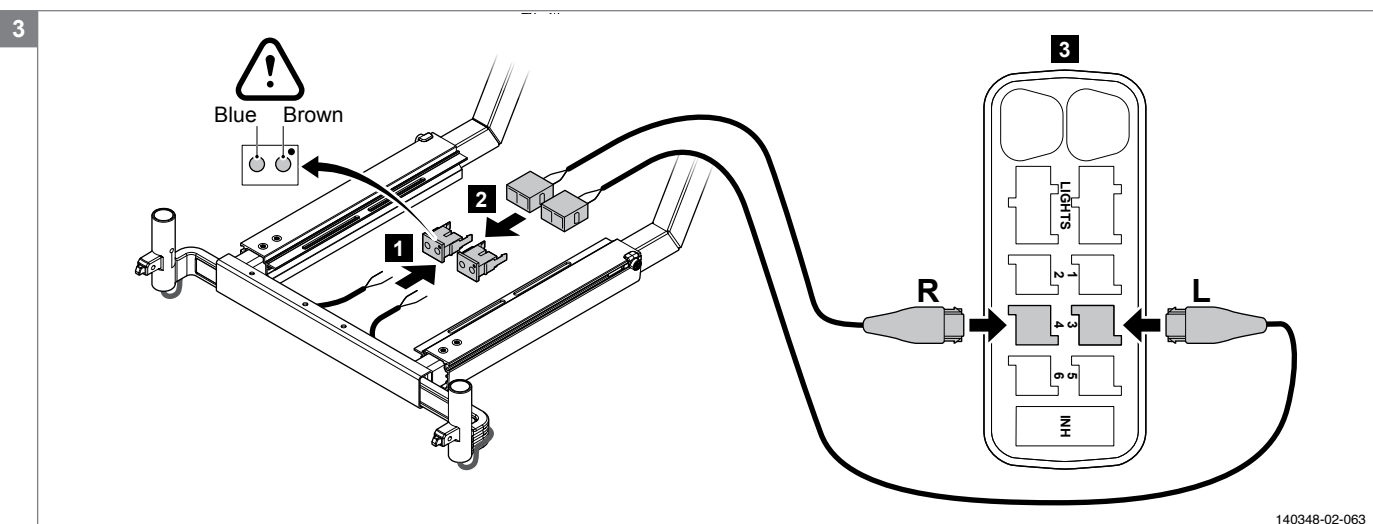
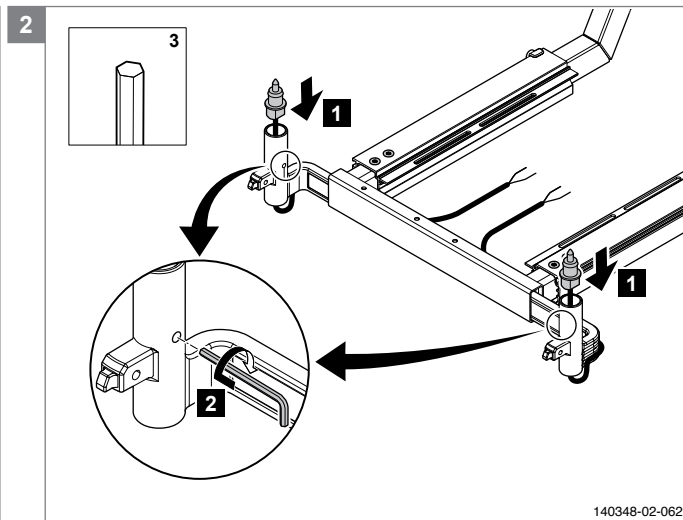
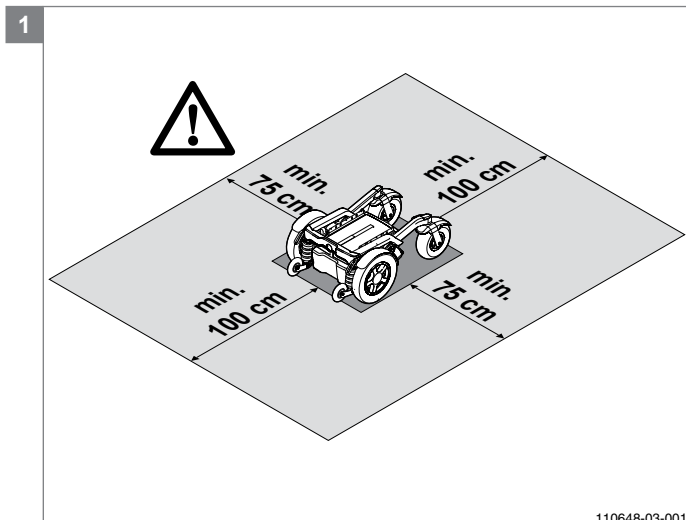
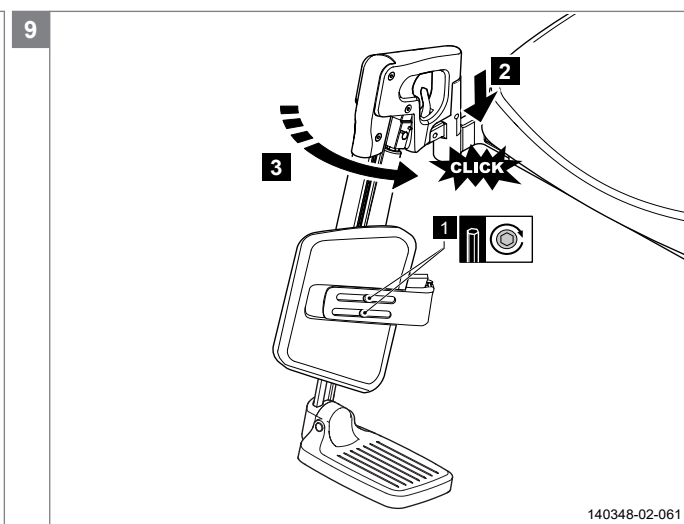
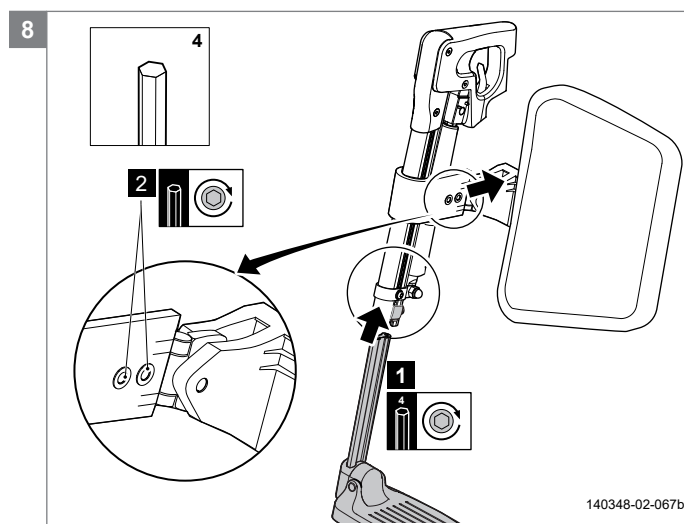
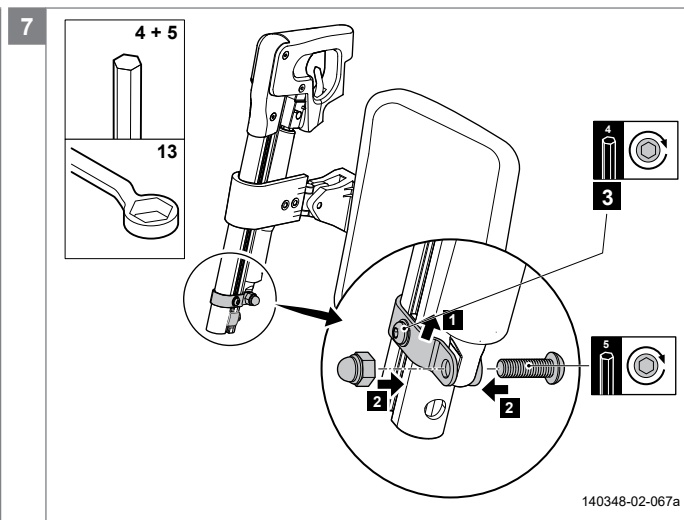
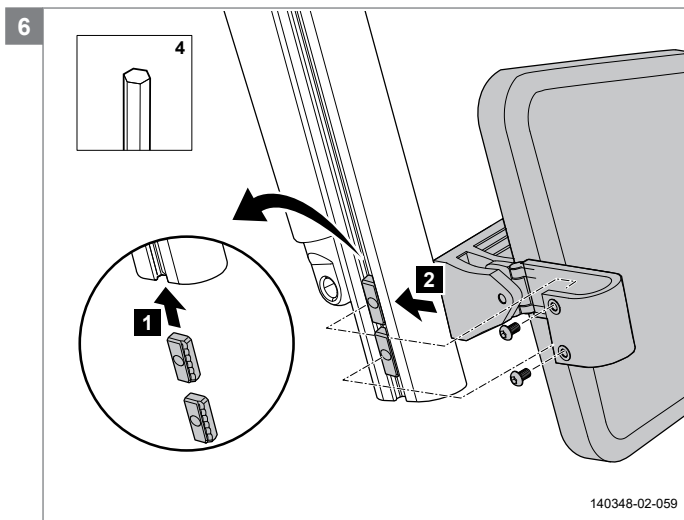
SEDEO

9010205C

Mounting the electrically elevating legrest

Preparation

Note! Switch off the wheelchair via remote control.

Instructions

Note! Suitable work area required!

- Remove seat and legrests.
- Place cable with plug from the top downwards into the tube from the legrest support. Secure the plug.
- Attach the connectors to the cables.

Note! Attach the brown cable to connector position with the (dot-) marking.

- Attach the cable extensions to the connectors and control box.

Note! See the backside of the control box cover for a wiring diagram.

- Release the footplate fixing screw and lower the footplate to remove it from legrest.
- Unscrew the lower clamp of the damper unit.
- Position the clamp nuts in legrest profile and attach the calf plate to the clamp nuts.
- Adjust the calf plate height to the desired position, tighten the clamp nuts.
- Remount the lower clamp of the damper unit.
- Reposition the footplate and tighten the fixing screw. Adjust the calf plate depth to the desired position.
- Adjust the calf plate inwards/outwards to the desired position. Lower the legrest into the tube of the mounting bracket, holding the footplate outwards. Swing the legrest inwards to lock it in position.

Note! Make sure the locking pin falls in place (with a 'click') and secures the legrest from becoming detached.

Note! If locking fails: Adjust the screw from the locking mechanism.

Relevant article numbers

1018306 Electrically elevating legrest set with calfplates SW 42-46

1018297 Electrically elevating legrest RHS SW 42-46 cm

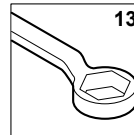
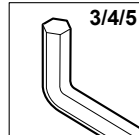
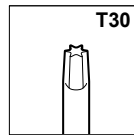
1018300 Electrically elevating legrest LHS SW 42-46 cm

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Sunrise Medical AB