


Hemiplegia armrest
Hemiplegiarmlenet
Hemiplegiarmstödet

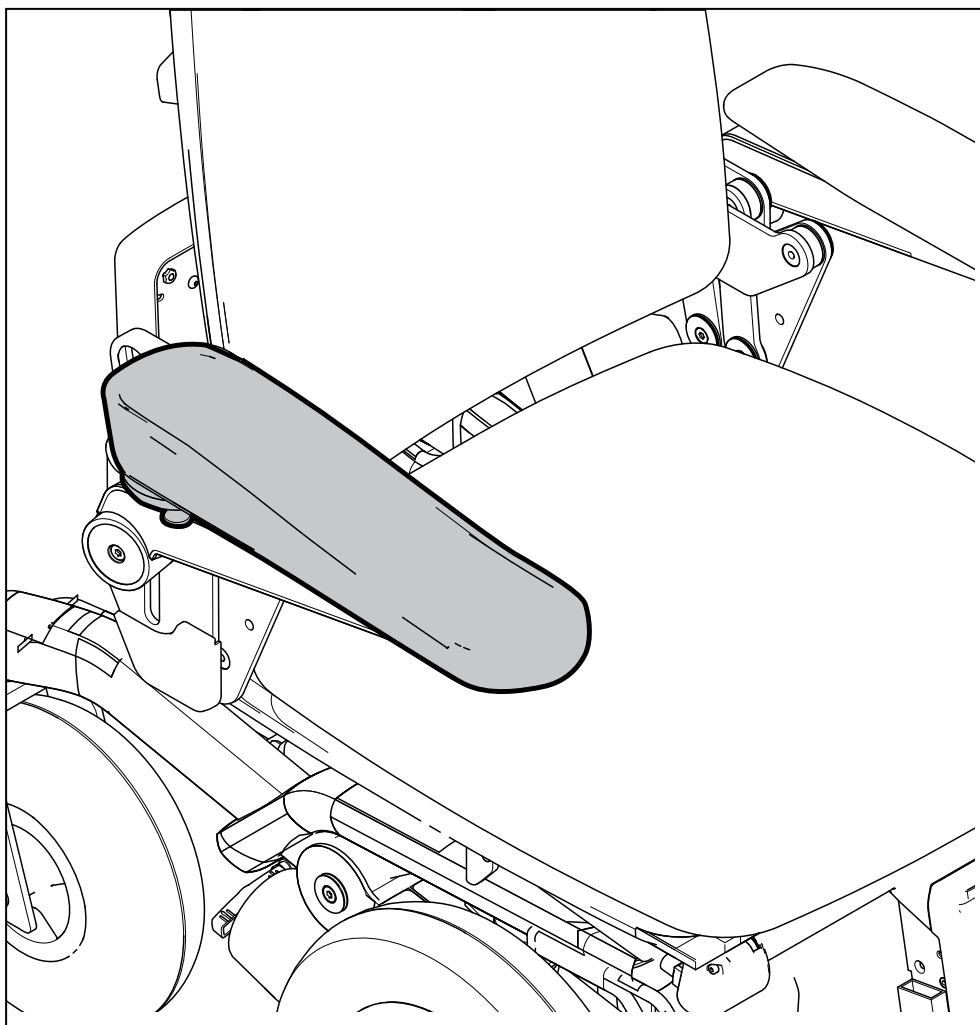

Sedeo Pro+





Hemiplegia armrest




ASSEMBLY INSTRUCTIONS
MONTERINGSANVISNING
MONTERINGSANVISNING

EN
NO
SV

SEDEO

9010214C

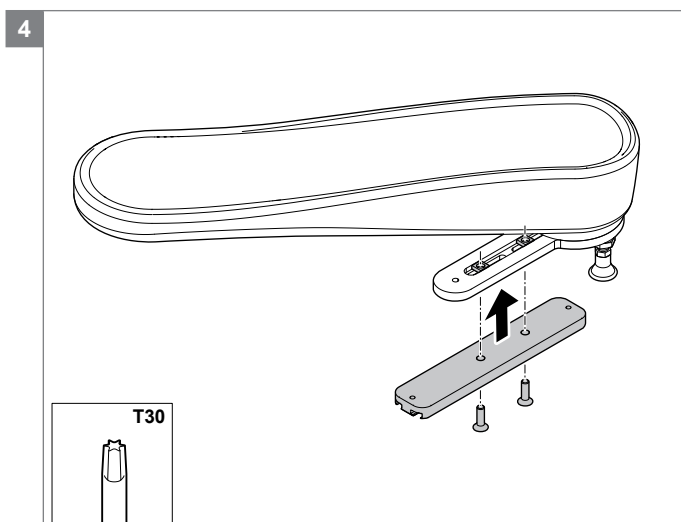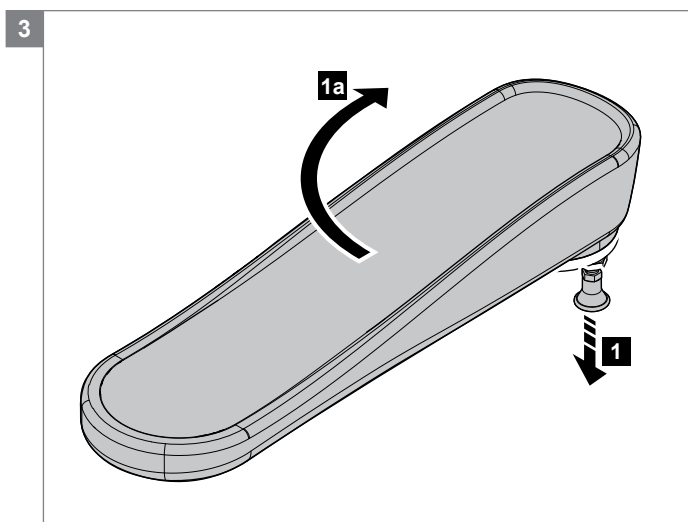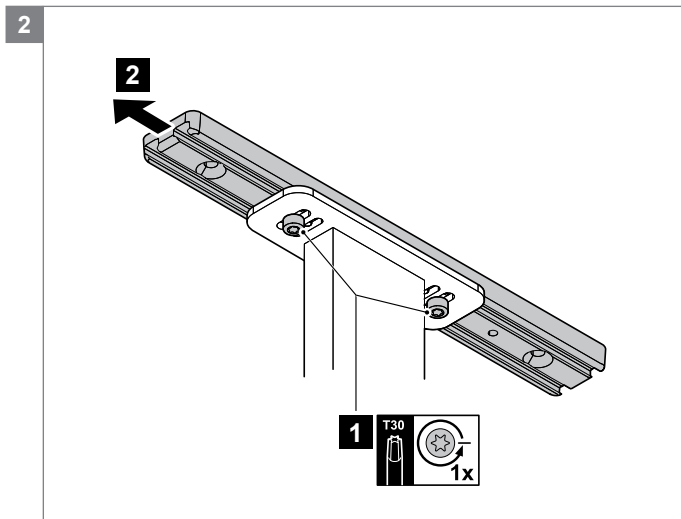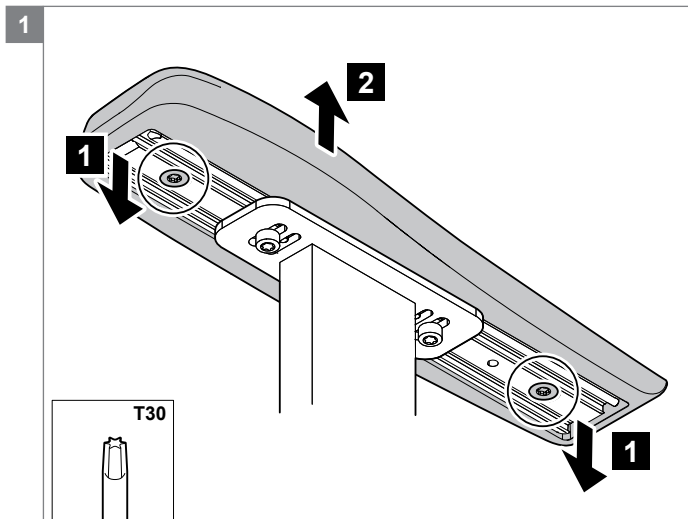

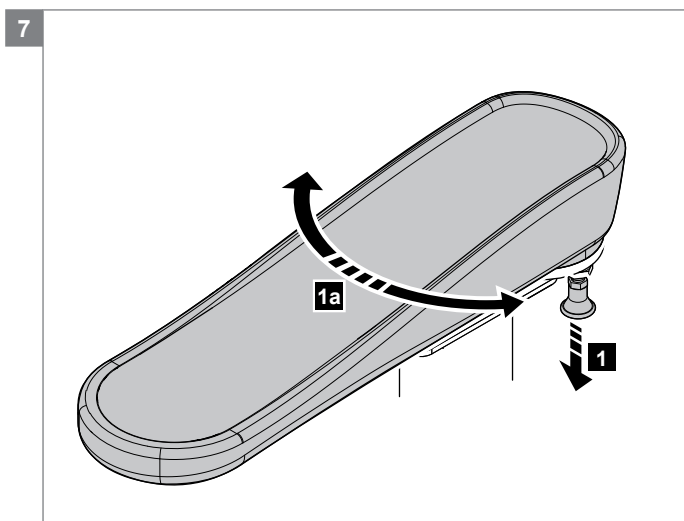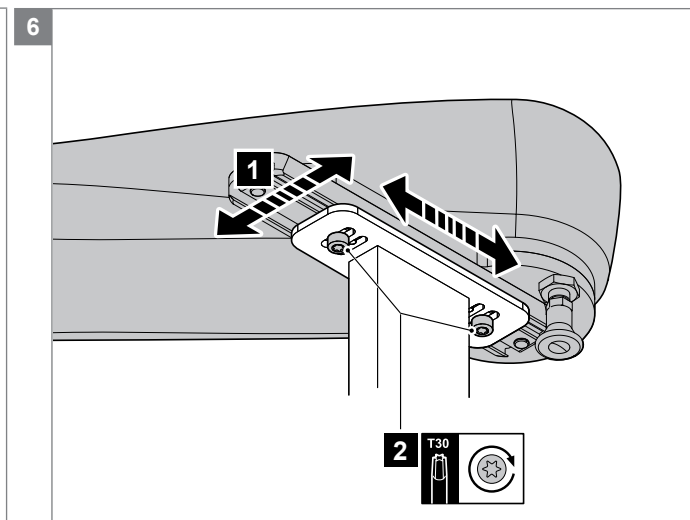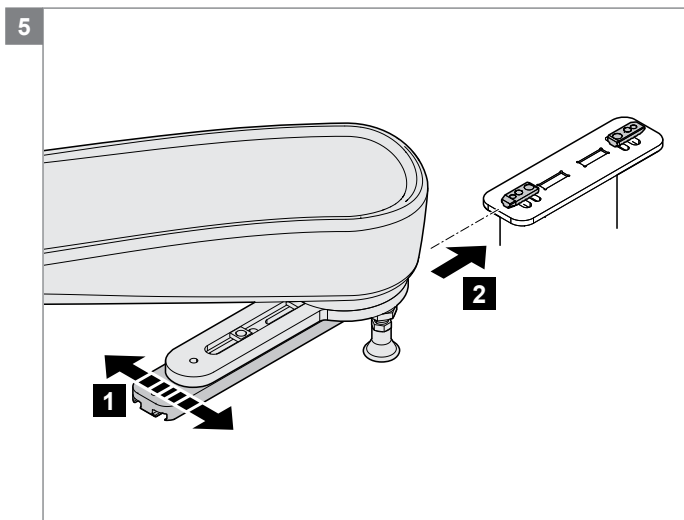

Mounting the hemiplegia armrest

Preparation

Note! Switch off the wheelchair via remote control.

Relevant article numbers


1010977 Hemiplegia armrest

Instructions

Note! Suitable work area required!

- Unscrew the armrest which is to be replaced from its mounting profile.
- Release the clamp nuts holding the mounting profile.
- Slide the mounting profile from the clamp nuts.
- Release the locking pin on the Hemiplegia rest and turn the pad sideways to expose the mounting bracket.
- Screw the mounting profile to the Hemiplegia mounting bracket.
- Slide the Hemiplegia armrest to the clamp nuts.
- Fasten the clamp nuts at the desired position.
- Swing back the Hemiplegia armrest and lock it in position.

Tools used

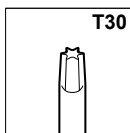

Montere hemiplegiarmlenet

Forberedelser

Merk! Slå av rullestolen via styreboksen

Relevante artikkelnumre


1010977 Hemiplegiarmlene

Anvisninger

Merk! Bruk et egnet arbeidsområde!

- Skru armlenet som skal skiftes løs fra festeprofilen.
- Løsne klemmemutterne som holder festeprofilen.
- Skyv festeprofilen løs fra klemmemutterne.
- Løsne låsepinnen på hemiplegiarmlenet og dreii paden til siden for å eksponere festebraketten.
- Skru monteringsprofilen fast i hemiplegifestebraketten.
- Skyv hemiplegiarmlenet inn på klemmemutterne.
- Fest klemmemutterne i ønsket posisjon.
- Sving hemiplegiarmlenet tilbake og lås det på plass.

Verktøy som brukes

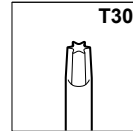

Montering av hemiplegiarmstödet

Förberedelser

Obs! Stäng av manöverboxens av- och påknapp

Anvisningar


Obs! Lämpligt arbetsområde krävs!

- Skruva loss det armstöd som ska bytas ut från monteringsprofilen.
- Lossa låsmuttrarna som fäster monteringsprofilen.
- Skjut loss monteringsprofilen från låsmuttrarna.
- Lossa låsstiftet på hemiplegiarmstödet och vrid polstret i sidled för att exponera fästkonsolen.
- Skruva fast monteringsprofilen på hemiplegifästkonsolen.
- Skjut fast hemiplegiarmstödet på låsmuttrarna.
- Dra åt låsmuttrarna vid önskat läge.
- Vrid tillbaka hemiplegiarmstödet och lås det i dess läge.

Relevanta artikelnummer

1010977 Hemiplegia armrest

Verktyg som krävs

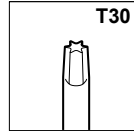

AU
Sunrise Medical Pty. Ltd.
6 Healey Circuit, Huntingwood
NSW 2148
Australia
T: +61 (2) 9678 6600
E: enquiries@sunrisemedical.com.au
www.SunriseMedical.com.au

BE
Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands
T: +32 (0)78 150 159
E: customerservice@sunrisemedical.be
www.SunriseMedical.be

CH
Sunrise Medical AG
Lückhalde 14
3074 Muri/Bern
Schweiz/Suisse/Svizzera
T: +41 (0)31 958 3838
E: info@sunrisemedical.ch
www.SunriseMedical.ch

DE / AT
Sunrise Medical GmbH & Co. KG
Kahlbachring 2-4
69254 Malsch/Heidelberg
Deutschland
T: +49 (0) 7253/980-0
E: kundenservice@sunrisemedical.de
www.SunriseMedical.de

DK
Sunrise Medical ApS
Park Allé 289 G
DK-2605 Brøndby
T: +45 70 22 43 49
Email: info@sunrisemedical.dk
www.SunriseMedical.dk

ES / PT
Sunrise Medical S.L.
Polígono Bakiola, 41
48498 Arrankudiaga – Vizcaya
España
T: +34 (0) 902142434
E: marketing@sunrisemedical.es
www.SunriseMedical.es

FR
Sunrise Medical S.A.S
ZAC de la Vrillonnerie
17 Rue Mickaël Faraday
37170 Chambray-Lès-Tours
T: + 33 (0) 2 47 55 44 00
E: info@sunrisemedical.fr
www.SunriseMedical.fr

IT
Sunrise Medical S.r.l.
Via Riva, 20 – Montale
29122 Piacenza
Italia
T: +39 0523 573111
E: info@sunrisemedical.it
www.SunriseMedical.it



NL
Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands
T: +31 (0)492 593 888
E: customerservice@sunrisemedical.nl
www.SunriseMedical.nl
www.SunriseMedical.eu (International)

NO
Sunrise Medical AS
Delitoppen 3
1540 Vestby
Norway
T: +47 66963800
E: post@sunrisemedical.no
www.SunriseMedical.no

PL
Sunrise Medical Poland Sp. z o.o.
ul. Elektronowa 6,
94-103 Łódź
Polska
T: + 48 42 275 83 38
E: pl@sunrisemedical.de
www.Sunrise-Medical.pl

SE
Sunrise Medical AB
Box 9232
400 95 Göteborg
Sweden
T: +46 (0)31 748 37 00
E: post@sunrisemedical.se
www.SunriseMedical.se

UK
Sunrise Medical
Thorns Road
Brierley Hill
West Midlands
DY5 2LD
United Kingdom
T: +44 (0) 845 605 66 88
E: enquiries@sunmed.co.uk
www.SunriseMedical.co.uk

Dealer / distributor:

Serial number:



MD_Sedeo_hemiplegia_armrest_EU_EN_NO_SE_RevC_2016_11_25

