
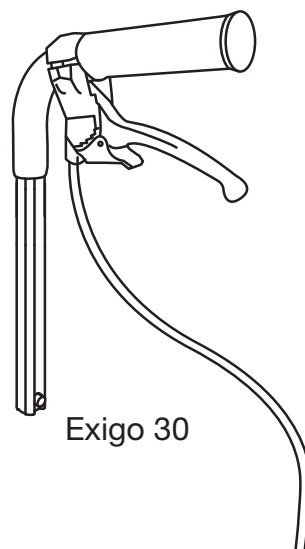
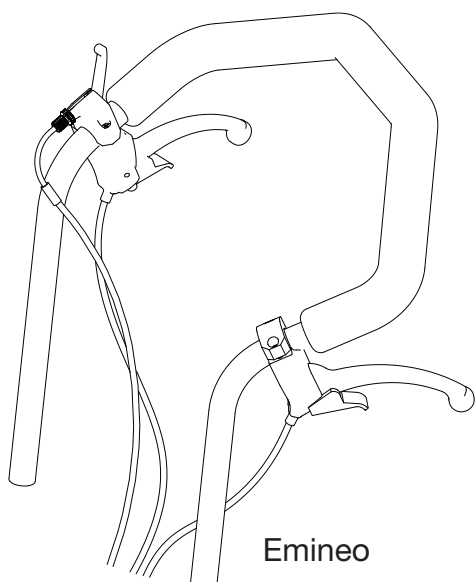
MONTERINGSANVISNING

Ledsagarbroms, One-tool






Benämning	Artikelnummer
20" Lufthjul	<input type="checkbox"/> 84416
22" Lufthjul	<input type="checkbox"/> 84436
24" Lufthjul	<input type="checkbox"/> 84456
20" Kompakta hjul	<input type="checkbox"/> 84419
22" Kompakta hjul	<input type="checkbox"/> 84439
24" Kompakta hjul	<input type="checkbox"/> 84458
Monteringsatts till ledsagarbroms One-tool	<input type="checkbox"/> 3409990

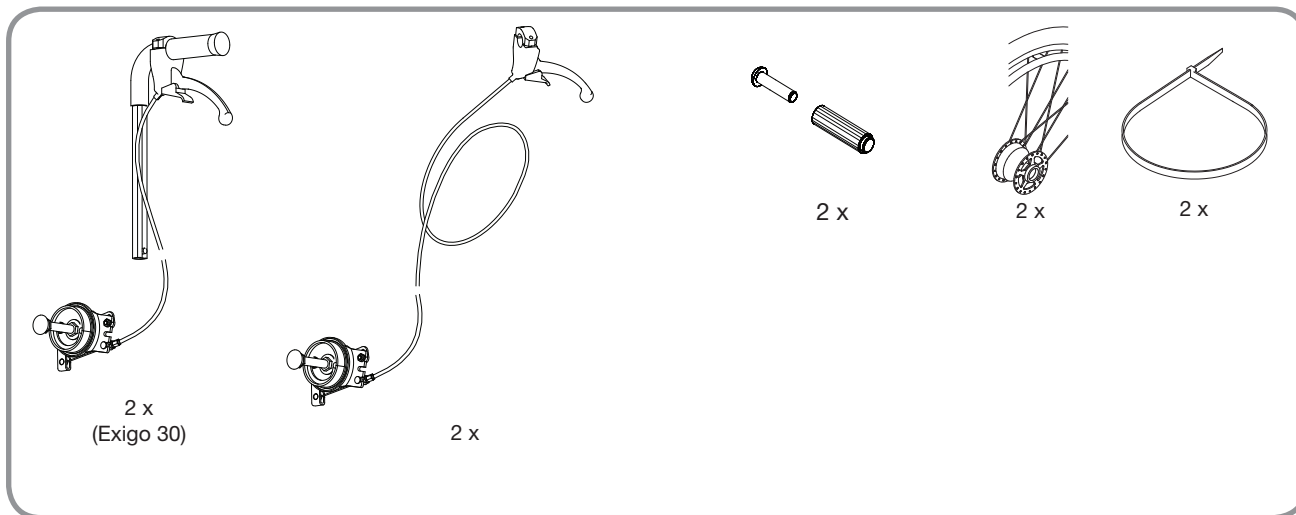 **Breezy**[®]

EN

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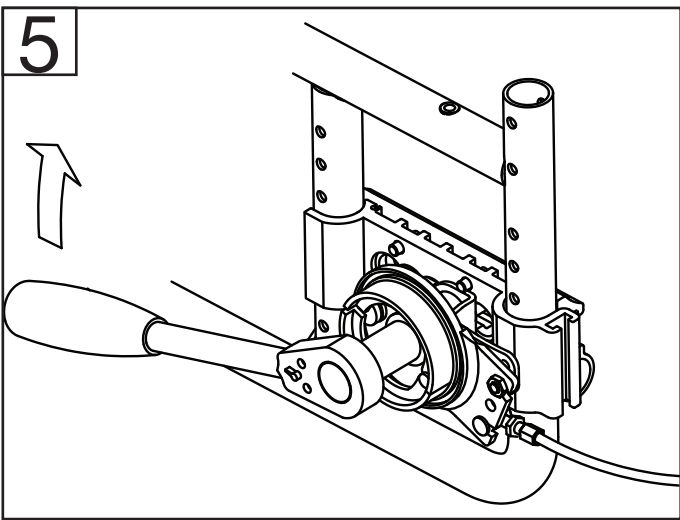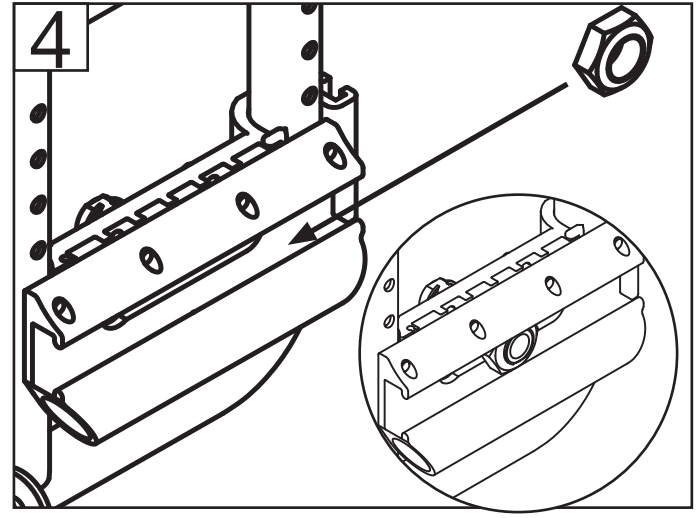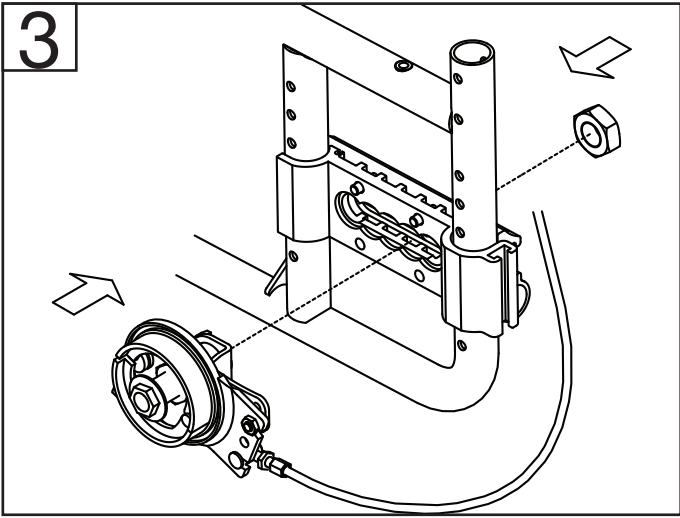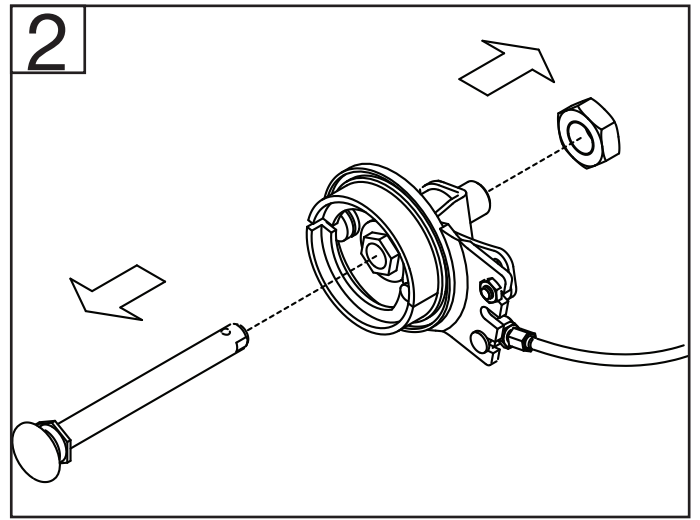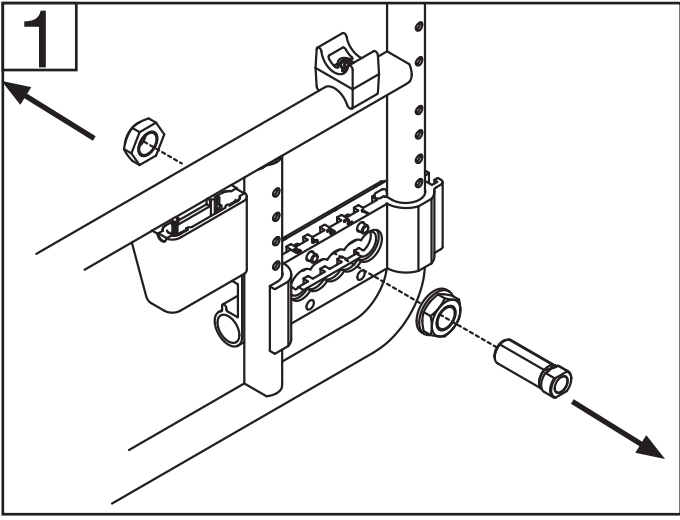6. a Brake lever Exigo 30
Dismount the old push handles and fit the new ones.
 - b Brake lever Emineo
Assemble the brake lever as displayed in figure 6b. Illustration in the circle displays assembling together with tilt lever.
 7. Dismount the old brake blocks and fit the new ones.
 8. To adjust the drum brake, screw out the wire stretcher step by step. Mount the wheel and test the brake by spinning the wheel and squeezing the brake handle. The wheel should spin easily.
- Use strips to fasten the brake-wire under the back cover.
9. Adjusting the seat height and the centre of gravity
Use a 17 mm socket wrench to loosen the casing. Turn the casing until you can loosen the wheel block.

10. Seat height

Pull the inner wheel block slightly apart from the frame. Adjust the wheel block step-by-step up or down.

Squeeze the inner and outer wheel block together. It is important to ensure that the pins in the wheel block go into the holes in the frame and that the casing lies horizontally in the track before tightening the screw.

Centre of gravity

Pull the casing slightly out and adjust forward or backward.

Squeeze the inner and outer wheel block together. It is important to ensure that the pins in the wheel block go into the holes in the frame and that the casing lies horizontally in the track before tightening the screw.

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Sunrise Medical HCM B.V.
Sunrise Medical AB