JAY® JAY 2 Plus Cushion

Important Consumer Information

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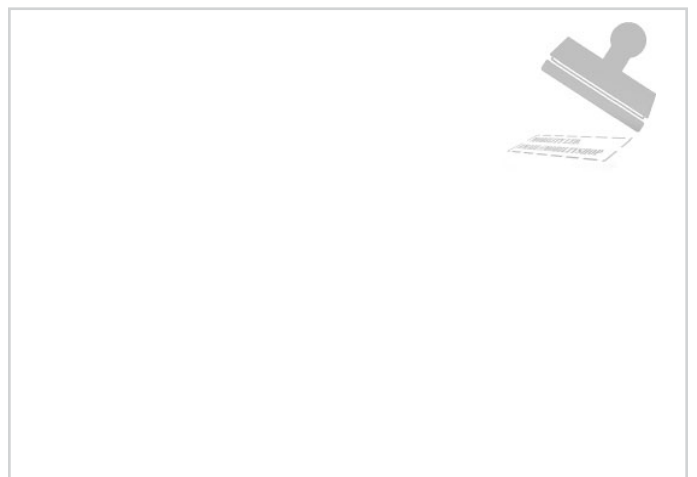
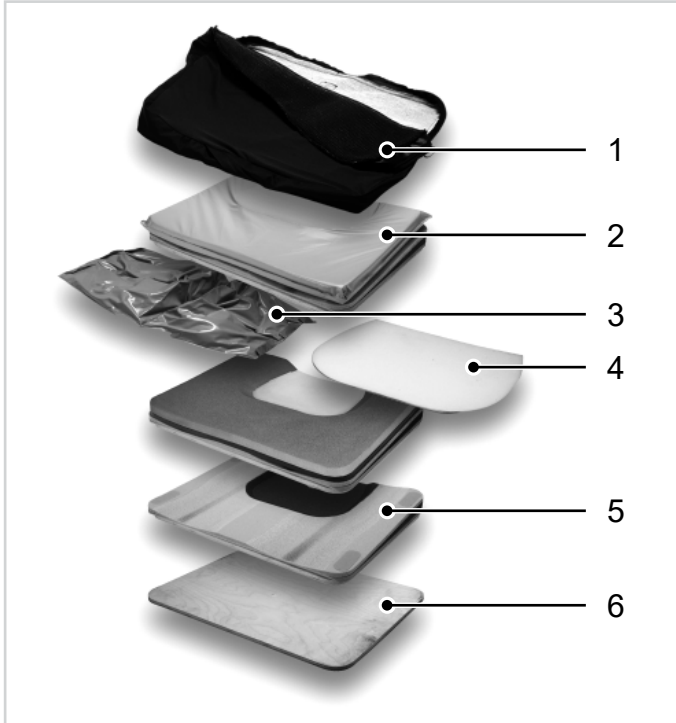
Bruksanvisning

The **J2 Plus Cushion** was developed for clinicians to use with clients needing a load capacity of up to 650 lbs who are at low to high risk for skin breakdown. Designed for excellent skin protection and postural stability.

THE J2 PLUS CUSHION COMPONENTS:

1. Cushion Cover
2. Urethane encased foam overlay pad
3. Fluid insert
4. Foam inserts (3 layers)
5. Pre-contoured foam base
6. Solid Seat (optional)

INSTRUCTIONS FOR J2 PLUS CUSHION

⚠ CAUTION

Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.

Installing a cushion on a wheelchair may affect the centre of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

The J2 Plus Cushion is designed to help reduce the incidence of skin breakdown on the sitting surface. However, no cushion can completely eliminate sitting pressure or prevent pressure sores. The J2 Plus Cushion is not a substitute for good skin care including proper diet, cleanliness, and regular pressure reliefs.

Do not leave the cushion outside overnight at temperatures below 40° F (5° C). Allow the cushion to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage.

Never submerge the cushion or any component other than the external cover in any liquid.

Avoid sharp objects or exposure to excessive heat or open flame.

Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. Replace stretched upholstery or purchase the J2 Plus Solid Seat Insert.

Avoid using overly stretched backrest upholstery. Stretched back upholstery does not provide adequate support for the body and improper positioning may result. Replace stretched backrest upholstery or purchase a back support product from Sunrise that is appropriate for you. Contact an authorized supplier for a free demonstration of these products.

When using this cushion with the fluid pad option, the fluid may lose volume over time. Volume loss in this cushion may result in bottoming out the cushion.

Monthly, during routine cleaning and inspection, check this cushion for bottoming out (see instructions under "checking for bottoming out" below) and any fluid inconsistencies. If the fluid is firmer in one area simply knead it back to its original consistency.

If the fluid has shifted, this may result in bottoming out. Make sure to move the fluid so it has even distribution.

CHECKING FOR BOTTOMING OUT ON THE J2 PLUS CUSHION

Bottoming out occurs on a J2 Plus Cushion when you displace the fluid out from underneath your seat bones, which leaves you sitting on the foam base. It sometimes occurs on very bony individuals, people using recliner wheelchairs, people who slouch when sitting, people who move a lot, shifting the fluid, or if using a cushion that is too wide.

When bottoming out occurs, increased pressure is placed onto the ischials and coccyx increasing the risk for skin breakdown.

To check for bottoming out, sit on the cushion without the cover for a minimum of two minutes. Transfer up and off the cushion (or have someone help you transfer), trying not to disturb the fluid underneath you. Push down in the depressions on the pad where your ischials (seat bones) and coccyx (tailbone) were. You should have to push through at least 1/2" (1cm) of fluid before you feel the firm cushion base below.

If the cushion is properly positioned and the footrests are properly adjusted, and there is not at least the minimum 1/2" (1cm) of fluid, the cushion is bottoming out and should not be used.

If bottoming out occurs discontinue use of this cushion and contact your healthcare professional. If further assistance is necessary, contact your Sunrise Medical Supplier.

FITTING THE J2 PLUS CUSHION

A healthcare professional should ideally fit a pressure-distributing cushion to you. Improper cushion fit may be detrimental to the performance of a pressure-distributing cushion and its support of the body.

Place the cushion to the back of the wheelchair with the material identification tags and fluid pad or the foam inserts toward the rear. When fit correctly, your hips should be to the back of the chair. Your ischials (seat bones) should be centered on the JAY Flow™ fluid pad or on the foam inserts in the seat well of the cushion. Your hip bones (trochanters) should be supported by the foam along the lateral (outside) borders of the well. Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures. Adjust the footrests so your legs rest firmly but comfortably on the cushion.

NOTE: We recommend using a J2 Plus Solid Seat Insert. It will optimize the performance of your J2 Plus Cushion, help provide better positioning and increase the life of the cushion.

ASSEMBLING THE SEAT WELL

To Insert Well Foam (Figure 1)

Stack the 3 layers of foam (foam well inserts) with smallest toward the bottom and largest on top. Be sure the beveled edges are in the same direction as the well cutout forming a single, angled edge. Fold all 3 layers of foam and insert into pocket until they fit fully into the well. Allow the foam to unfold so that the rounded corners fit snugly into the back of the pocket.

To Insert Fluid Pad (Figure 2)

Fold fluid pad in half so that Velcro® is on the outside. Slide Fluid Pad into pocket, making certain that the back of the pad is toward the opening of the envelope. The word "Back" is printed on a tag on the Fluid Pad. Position the pad to the correct side of the pocket and unfold. Be certain that Velcro on the Fluid Pad mates to Velcro in the urethane pocket.

To Reseal the Urethane Envelope (Figures 3 & 4)

Position urethane envelope with dark gray side up. Tuck dark gray layer between insert foam and well pocket. Folding left and right sides in at 45° angles will form a smooth flap (Fig. 3). Fold the light gray flap around entire cushion thickness and fasten by aligning 3 Velcro strips (Fig. 4).

EASY MAINTENANCE AND CLEANING

Monthly cleaning and regular maintenance may help extend the life of your cushion

To Clean Cover

Remove the cover and turn it inside out. Close the zipper and machine wash on warm cycle and drip or tumble dry on low heat. Do not use industrial dryers. Do not steam autoclave. Do not bleach.

NOTE: When the cover is removed from the cushion for cleaning, inspect the fluid pad or foam inserts and foam base for unusual wear. Feel the fluid for any inconsistencies. If inconsistencies are found, knead the fluid pad to original consistency and/or call your Sunrise Medical supplier.

To refit cover after washing, reinstall any accessory items or fluid / foam inserts, then fit the cover back over the foam base and urethane encased foam overlay. Ensure the back of the cover matches up to the back of the foam base.

Use only JAY designated covers on JAY products

To Clean Foam Base

Remove the cover, urethane encased foam overlay pad, and any accessory parts from the foam base. Wipe with warm water and soap. Wipe off with a clean cloth.

NOTE: Avoid submersion of the foam base in water

Reinstall the urethane encased foam overlay pad, and any accessory parts, then fit cover back over complete assembly. Ensure the back of the cover matches up to the back of the foam base.

To Clean the Urethane Encased Foam Overlay Pad

Remove the cover from the complete assembly. Remove the overlay and wipe with warm water and soap. Wipe off with a clean cloth.

NOTE: Do not submerge in water. Water may be absorbed into the foam through the air vents on the top of the pad or through the rear Velcro closure.

Reinstall overtop of the foam base, and fit the cover back over the complete assembly.

Ensure the back of the cover matches up to the back of the foam base.

Fig. 1

Fig. 2

Fig. 3

Fig. 4

WARRANTY

Each JAY cushion is carefully inspected and tested to provide peak performance. Every JAY cushion is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

There is no implied warranty beyond what is contained herein.

Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

JAY FRONT ↑

Part Code: _____

Date: _____

cm kg Max

Sunrise Medical (US) LLC
2842 Business Park Avenue
Fresno, California, 93727 USA

EC REP Sunrise Medical GmbH
Kahlbachring 2-4
69254 Malsch / HD
Germany
Assembled in Mexico

Cover Materials:
Polyester
Urethane
Vinyl

THIS SIDE DOWN

BACK ↓

248676 Rev. B

1	Type:	Product Name	CE	CE Mark
2	Date:	LOT: defined by Julian Date	XXXX-XX-XX	Date of Manufacture
		Seat width	Part Code	Assembly Part Number
		Seat depth	MD	This symbol means Medical Device
		Max user weight	Manufacturer	EC REP
				European Authorised Representative

We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R & D to production. This product meets the requirements in accordance with EU regulations. Options or accessories shown are available at extra cost.

The varieties of cushions fitting variants, as well as the different positioning components to deliver higher positioning and skin protection benefits according to each user requirement and indication, mean that it can be used by a range of users with different positioning and skin protection needs and indications as follows[R7-R14] :

- Spinal Cord Injury
- MS
- Neurological disease pattern (hemiplegia, M. Parkinson)
- Extremity defect deformity
- Joint contracture
- Amputation
- Tetraplegia

The cushion for a wheelchair shall not be used in case of [R3, R4]:

- Redness developed in the skin contact area
- Bottoming out present in the JAY Fluid Insert
- Decubitus

NOTE: General user advice. Not following these instructions may result in physical injury, damage to the product or damage to the environment!

A notice to the user and/or patient that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

B4Me special adaptations

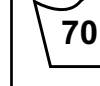
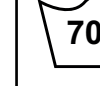
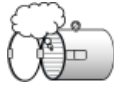
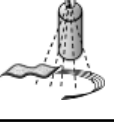
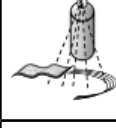
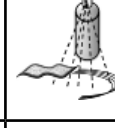
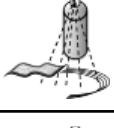
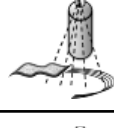
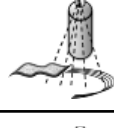
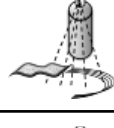
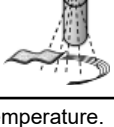
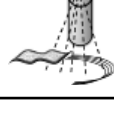
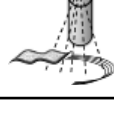
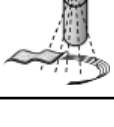
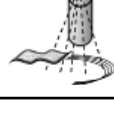
Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used. Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.Sunrisemedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, section 14.1 of the Medical Device Regulation 2017/745.

Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk.

Approved Cleaning Procedures for JAY Cushions

Product	JAY Lite	JAY J3	J2 GS	JAY Easy Visco	JAY Soft Combi P	JAY Xtreme Active	JAY Basic	JAY Easy	JAY Balance
Outer Cover (Microclimatic and Stretch)									
(1). Wet Wash 70°C 									
Outer Cover (Incontinent)									
(2). Wet Wash 60°C 									
Inner Cover									
(3). Spray and Wipe disinfection  or Wet Wash 60°C	NA	NA	NA	NA	NA		NA	NA	
Cushion core (foam base)									
(4). Steam autoclave up to 105 °C (1) 	✗	✗	✗	✗	✗				✗
Spray and wipe disinfection 						✗	✗	✗	✗
(5). Immersion disinfection 				✗	✗	✗	✗	✗	✗
Cushion elements									
Roho Air element	NA			NA	NA	NA	NA	NA	
Positioning elements	NA			NA	NA		NA	NA	
Fluid pad	NA			NA	NA		NA		

1. 70°C is the maximum recommended temperature.

2. 60°C is the maximum recommended temperature.

3. The Xtreme Active and Balance inner covers can be washed at 60°C as well as spray and wipe disinfected. Possible discoloration if washed at 60°C but the functional aspect of the covers is not diminished.

4. An autoclave is a self locking machine that sterilizes with steam under pressure.

5. At low temperature (< 30°C)

The product warranty shall not apply to products damaged due to the failure to follow these cleaning instructions or the use of inappropriate cleaning compounds.

Recommended Disinfectants

All disinfectants which are listed and recommended by the Robert Koch Institute: http://www.rki.de/EN/Home/homepage_node.html

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S