JAY[®] ZIP[™] Back

Important Consumer Information

NOTE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

WARRANTY REGISTRATION: To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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⚠ WARNING

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a ZIP Back is appropriate. Backs should only be installed by an authorized Sunrise Medical supplier.

THE JAY ZIP Back (Fig.1.0)

The JAY ZIP Back is designed to provide the important benefits of a proper back support for a wide variety of pediatric users.

It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is light-weight, durable and easy to use.

The intended lifetime of this product is 5 years.

Maximum user weight: 75kg

⚠ WARNINGS

- Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury.
- Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.
- Do not use the back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.
- Do not use the back handle to lift wheelchair. It is intended only for back removal.

Type:	Product Name/ SKU Number	XXXX-XX-XX	Date of Manufacture
Date:	LOT: defined by Julian Date	Part Code	Assembly Part Number
 XXX mm	Seat width	MD	This symbol means Medical Device
 XXX mm	Seat depth		Manufacturer's address
 XXX kg	Max user weight	EC REP	European Authorised Representative
CE	CE Mark	UK RP	UK Responsible Person
UK CA	UKCA Mark	CH REP	Swiss Representative's address
	Consult instructions for use		Importer's address

SAMPLE

JAY

Part Code:

Date:

cm cm kg

Max

Sunrise Medical (US) LLC
2842 Business Park Avenue
Fresno, California, 93727 USA

EC REP Sunrise Medical GmbH
Kahlbachring 2-4
69254 Malsch / HD
Germany

UK RP Sunrise Medical Ltd
Thoms Road, Brierley Hill
West Midlands DY5 2LD
United Kingdom

Assembled in Mexico

106132 Rev. J

UK	RP	Sunrise Medical Limited Thorns Road, Brierley Hill West Midlands, DY5 2LD UNITED KINGDOM
-----------	-----------	---

The management system of SUNRISE MEDICAL is certified to EN ISO 13485 and ISO 14001.

We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R & D to production. This product meets the requirements in accordance with EU and UK regulations. Options or accessories shown are available at extra cost.

The varieties of backrest fitting variants, as well as the different positioning components to deliver postural support, pressure distribution, increase seating tolerance are benefits according to each user requirement and indication, mean that it can be used by a range of users with different positioning needs and indications as follows:

- Spinal Cord Injury
- MS
- Neurological disease pattern (hemiplegia, M. Parkinson)
- Muscle Dystrophy
- Amputation
- Tetraplegia
- Joint contracture
- Cerebral Palsy
- Extremity defect deformity

No contraindications are known or available.

NOTE: General user advice. Not following these instructions may result in physical injury, damage to the product or damage to the environment!

Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

CE As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

UK CA As the manufacturer, SUNRISE MEDICAL, declares that the product conforms to the UK Medical Devices Regulation 2002 No. 618.

B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used. Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.Sunrisemedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, section 14.1 of the Medical Device Regulation 2017/745. Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk.

MAINTENANCE

- Sunrise recommends that all fasteners be checked for wear, such as loose bolts or broken components every 6 months.
- Loose fasteners should be tightened according to the installation instructions.
- All fasteners should be tightened to torque specifications as stated in the hardware installation section.
- Contact your authorized Sunrise Medical supplier immediately to replace any broken components.

⚠ WARNING

- Do not continue to use the system after identifying loose or broken components.

NOTE: A wrench has been included with the back for your convenience.

EASY CLEANING (Fig.2.0)

⚠ CAUTION

When disassembling the back for cleaning, note the position of the Velcro® Lumbar Positioning Components for correct reassembly.

To clean back cover

Remove cover by detaching Velcro.
 Detach cover from back shell.
 Locate zipper on bottom of the cover and unzip.
 Remove foam with inner cover and turn cover inside out.
 Zip cover closed to avoid snagging.
 Machine wash in warm water (60°C).
 Drip or tumble dry on low heat.

⚠ CAUTION

- Do not dry clean covers or use industrial washers and dryers to clean cover.
- Do not steam autoclave.
- Do not bleach.
- Reinsert the foam with inner cover into the cover.

NOTE: The base has two layers of foam. When reinserting the foam, ensure that the thinner, firmer, light grey foam is facing the shell and the softer foam is facing the user.

Once the foam is secured inside the cover, zip cover closed.

To clean back shell and Inner Cover

Shell and inner cover may be wiped clean with soap and water.
 No other maintenance should be required.

To clean positioning components

Positioning components can be wiped clean with soap and water.

⚠ CAUTION

- Do not immerse positioning components underwater.
- If you have any additional questions regarding cleaning procedures, please contact your authorized Sunrise Medical dealer.

HYGIENE MEASURES WHEN BEING RE-USED

Prior to the wheelchair being re-used, it must be carefully prepared. All surfaces which come into contact with the user must be treated with a disinfection spray.

To do this, you must use a disinfectant as authorised/ recommended in your country, for rapid alcohol-based disinfection for medical products and medical devices, which must be disinfected quickly.

Please be aware of the manufacturer's instructions for the disinfectant you are using.

In general, a complete disinfection cannot be guaranteed on seams. We therefore recommend that you dispose of seat and back slings to avoid micro-bacterial contamination with active agents according to your local infection protection law.

CONDITIONS FOR SAFE STORAGE

- Products should be stored in a dry place: at 20 – 75% Relative Humidity.
- Protect from direct sunlight and dust for long periods.
- Away from direct heat and at a temperature between 5°C (40°F) to 40°C (104°F)
- Position and stack the packaged products according to the “this side up” arrows on the packaging.
- If a package is high and narrow and therefore potentially unstable, ensure it is well secured on the pallet or in the racking, to prevent it from falling down.
- Do not stack anything on top of the packaged products when the “do not stack” mark on the packaging is shown.
- Product should never be stored outside and subject to the elements.

FIG.2.0	LAUNDRY CARE SYMBOLS
	Machine wash in warm water (60° C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	Do not dry clean

DETERMINING COMPATIBILITY

Wheelchair Types

The ZIP Back is designed to be compatible with most wheelchairs with the following exceptions:

⚠ WARNING

- Wheelchairs that are angularly adjustable, recline or tilt and result in a back angle greater than 60 degrees from the vertical should not be used.
- Use with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair resulting in a fall and the potential for serious injury.

Wheelchair Dimensions

The JAY ZIP Back is designed to replace the wheelchair's sling upholstery.

Prior to installation determine if the wheelchair used has compatible back canes, by measuring the width between your back canes as seen in Figure 3.0.

Using that measurement, check Figure 4.0 to determine if the back will fit your chair.

Next check that the back canes are a compatible diameter.

The ZIP Back hardware can be mounted on wheelchair back canes that measure in diameter 3/4", (19mm) to 1 1/8" (29 mm). Measure your back cane diameter (Fig. 5.0) and refer to Figure 6.0 to ensure your back canes are compatible.

⚠ CAUTION

If you cannot determine that the back canes are compatible, please contact your authorized Sunrise Medical provider or Sunrise Medical customer service.

FIG.3.0

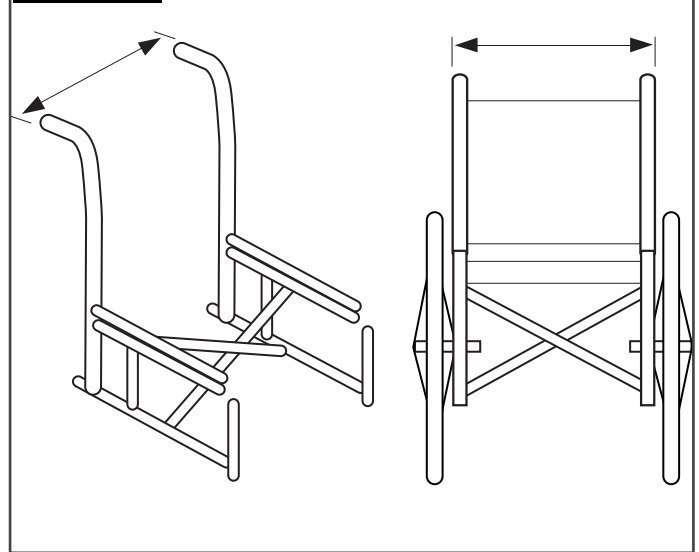
FIG.4.0

Width of back canes from outside to outside of the tubes (Fig. 3.0)

ZIP Back Width* (all contours)	Minimum width		Maximum width	
	Inch	mm	Inch	mm
8" (200mm)	7.7	200	9.3	240
10" (250mm)	9.7	250	11.3	290
12" (300mm)	11.7	300	14.3	360
14" (350mm)	13.7	350	16.3	410

FIG. 5.0

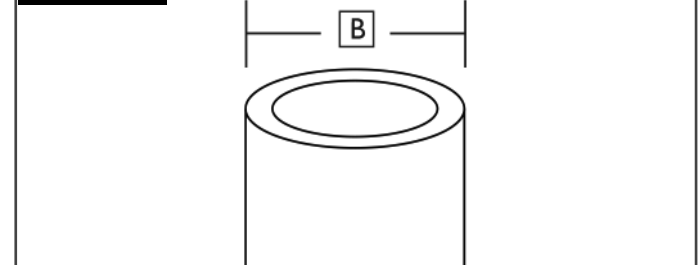
FIG.6.0

Cane Diameter (B)

ZIP Back Width* (all contours)	Minimum width		Maximum width	
	Inch	mm	Inch	mm
8" (200mm)	7.7	200	9.3	240
10" (250mm)	9.7	250	11.3	290
12" (300mm)	11.7	300	14.3	360
14" (350mm)	13.7	350	16.3	410

INSTALLING THE MOUNTING HARDWARE

Please read the following instructions before beginning installation.

To install the JAY ZIP Back, begin without the user in the wheelchair.

Once the back has been installed and minor adjustments are needed the user can be seated back in the wheelchair.

Tool required (included)

10mm wrench

4mm Hex Key

Hardware Installation Instructions (Fig 7.0-8.0)

- Remove the existing wheelchair back according to the wheelchair manufacturer's instructions.
- The bracket should come with mounting brackets attached (Fig. 7.0/A).
- Visually determine the desired location of the receivers on the wheelchair's back canes (Fig. 7.0/B).
- The mounting hardware should be at approximately equal heights on each back post and parallel to the seat frame (Fig.8.0).
- Higher locations on the back canes are generally better.

Attach the hardware receivers (Fig.9.0).

- Loosen the clamp screws (C) with the enclosed 4mm Hex Key. NOTE– For large tube diameters, clamp screws may have to be removed completely.
- Attach and align receiver to the wheelchair back canes (D).
- Hand tighten clamp screws (A).





NOTE: Receivers may require additional lateral adjustment to ensure proper alignment.

Attaching and Aligning the Back Shell (Fig 10.0)

- Loosen bracket nuts (E) and mounting pin bolts (F) using 10mm wrench until hardware can move easily in all directions.
- Attach the ZIP Back to the hardware receivers by inserting left and right mounting pin (F) in the left and right receivers (G).
- Locate back to the approximate location, relative to both chair and user.
- Tighten bolts on the inside of the mounting pin (F).
- The recommended torque specification is 100 - 110 inlbs. (11.3 - 12.4 Nm). NOTE– Back depth and angle may require additional adjustment to ensure proper fit to the user.

- Once properly aligned, tighten the hardware receiver clamp screws.

NOTE: The recommended torque specification is: 75 - 85 in-lbs (8.5 - 9.3 Nm).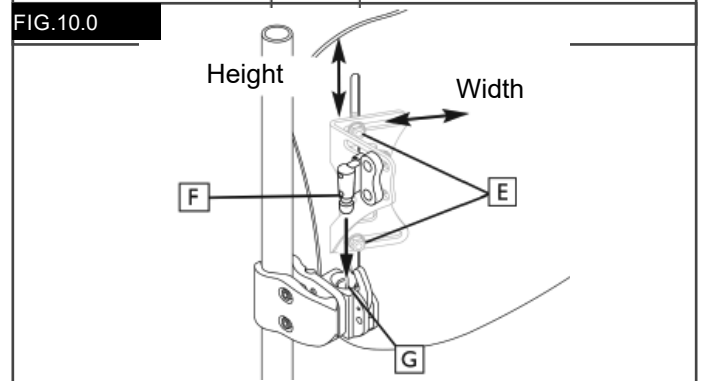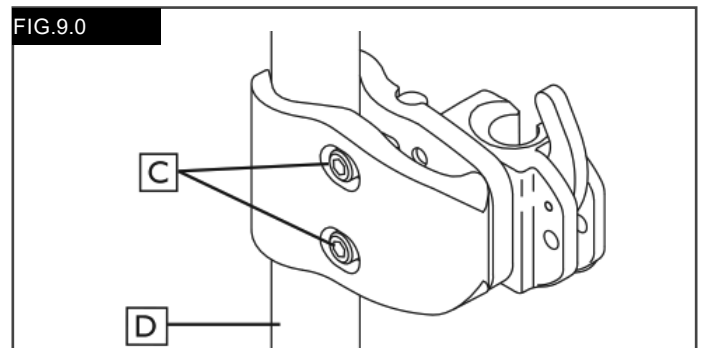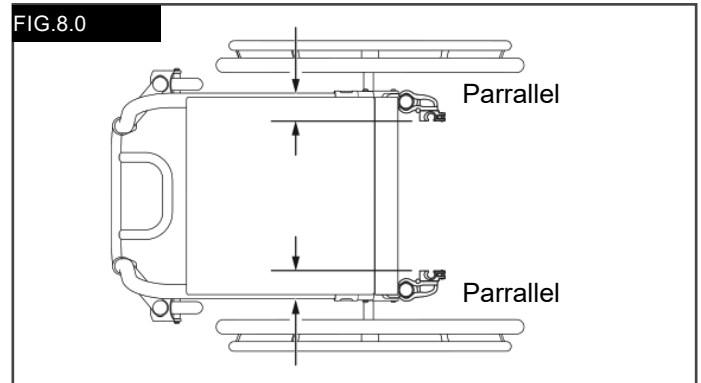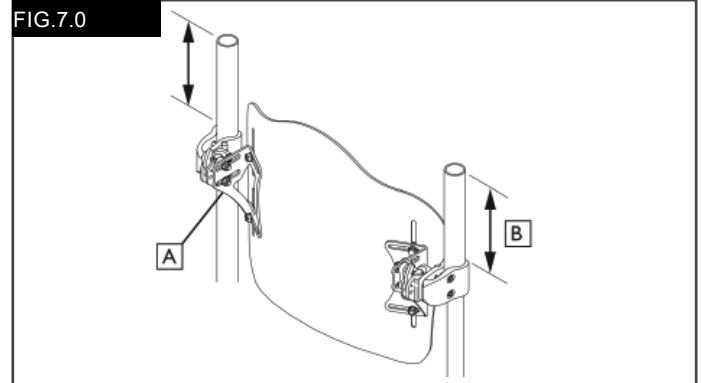

ADJUSTING JAY ZIP HEIGHT, DEPTH AND ANGLE

With the user in the chair, modify the back height, depth, and angle to meet the unique requirements of the user.

⚠ WARNING

- Do not have the user apply full pressure on the back as it is not fully tightened to the full torque requirements.

Height Adjustment (Fig. 11.0)

- With the user in the chair, identify the desired height of the back.
- Once identified, have the user remove pressure from the back by leaning or sliding partially forward if possible to allow the adjustment to take place.
- Loosen the bracket nuts (I) and adjust the height of the back shell to required position.
- Securely tighten the bracket nuts (I)
- The recommended torque specification is 75-85 in-lbs. (8.5 - 9.3 Nm).

NOTE: After height adjustment is complete, check that the width was not inadvertently changed when tightening the nuts.


Depth/Angle Adjustment

The hardware allows for 2", (50mm) of anterior depth adjustment, 2.75", (70mm) with reversing hardware.

- To adjust back depth and angle loosen left and right bolts (J) on bracket slots.
- Adjust depth and/or angle and securely tighten.
- The recommended torque specification is 100-110 in-lbs. (11.3 - 12.4 Nm).
- Ensure that none of the depth adjustment nuts (J) will slip or rotate.

⚠ WARNING

Ensure that all hardware has been properly tightened when all adjustments have been completed and that the Back is ready for use.

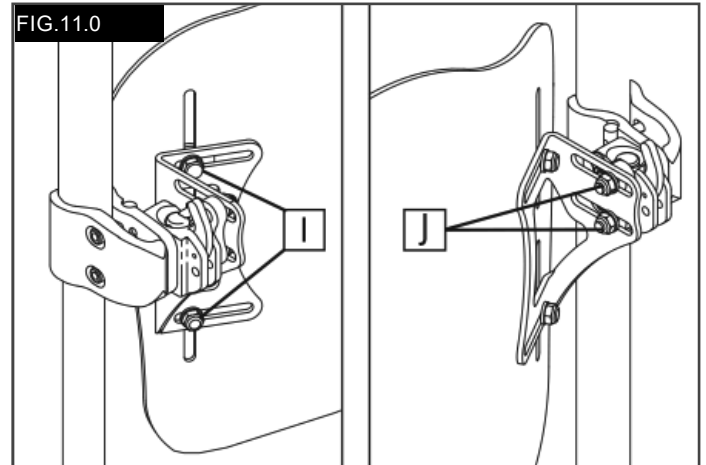BACK TRANSIT SAFETY

This back has been dynamically tested for use on a wheelchair transported in a motor vehicle. Please follow all installation, use and maintenance instructions within this manual as well as the transit instructions listed below.

⚠ WARNINGS

- If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the vehicle restraints.
- The distance between the top of the user's shoulder and the top of the back shell should not exceed 3" (7.6 cm).
- The wheelchair must be labelled as appropriate for use as a seat in a motor vehicle, dynamically tested to the performance requirements of ISO Standard 7176-19 and installed, used and maintained according to the manufacturer's instructions.
- If the original wheelchair seat was replaced, the new seat must be approved for wheelchair transit, installed and used as indicated in the manufacturer's instructions.
- The Wheelchair Tie down and Occupant Restraint System (WTORS) should be compatible with the specific wheelchair, used as indicated in the manufacturer's instructions and should comply with the performance requirements of ISO Standard 10542.
- The wheelchair must be forward facing during transport.
- In order to reduce the potential of injury to vehicle occupants wheelchair-mounted accessories such as trays and non essential equipment should be removed and secured separately.
- This product is intended for use by users ranging in weight from 10 to 75kg.
- Postural support devices such as pelvic positioning belts, anterior trunk supports or postural supports such as lateral trunk supports should not be relied on for occupant restraint in a moving vehicle unless they are labelled as conforming to ISO Standard 16840-4.
- Alterations or substitutions should not be made to the structure, parts or components, without consulting the manufacturer.
- **FAILURE TO HEED THESE WARNINGS COULD CAUSE SEVERE INJURY TO THE WHEELCHAIR RIDER OR OTHERS.**

REMOVAL AND REPLACEMENT**Removal**

Following installation and adjustment, the back can be easily removed from the wheelchair if necessary. To remove, pull down on back handle, rotate up, then remove from the hardware receivers.

Proper reattachment after removal

To reattach the back to the hardware receivers, pull down on cord, insert back into receivers, and rotate down.

⚠ WARNING

If you are uncertain if you have heard the locking "click", pull up on the back to ensure that it has been securely locked.

24 MONTH LIMITED WARRANTY

Each JAY ZIP Back is carefully inspected and tested to provide peak performance.

Every JAY ZIP Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

This warranty does not apply to punctures, tears or burns, nor to the back's removable cover. Claims and repairs should be processed through the nearest authorized supplier.

Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

There are not warranties which extend beyond the description on the face hereof.

Remedies for breach of express warranties herein are limited to repair or replacement of the goods.

In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S