

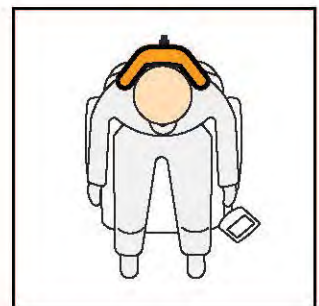
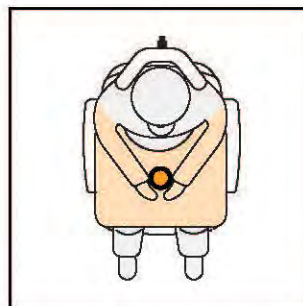
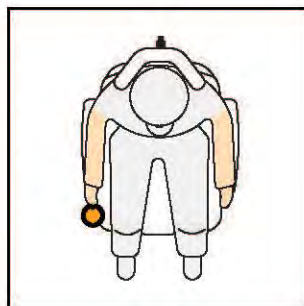
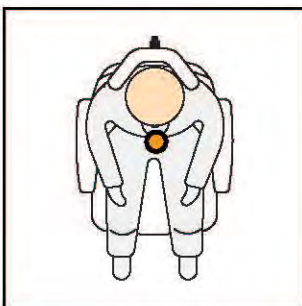


## Service manual Special Controls

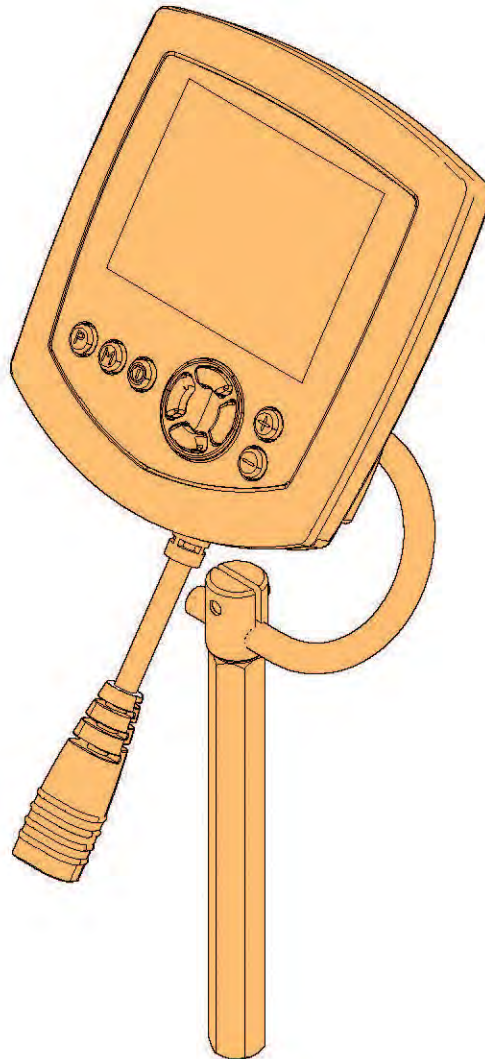


**Spesialstyringer  
monteringsanvisnig**

## Icon explanation

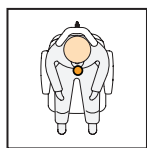

Chin Control

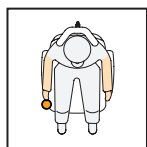

Hand Control


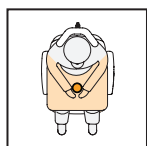



Table Top Control



Head Control

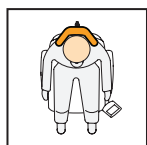


Omni

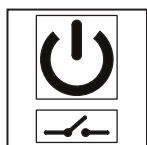

SW 0/1 (on /off switch)




SW "M" (mode/function switch)

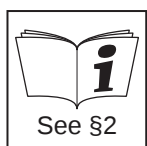


See manual




Safety instruction / Warning

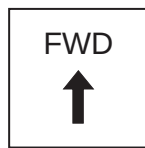


Forward

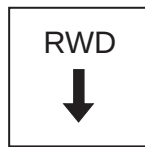


Backward

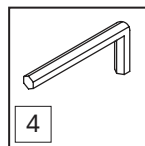


Allen Key

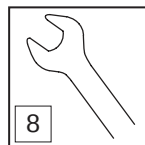


Wrench

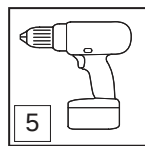


Drill

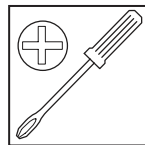


Screwdriver  
Phillips / Pozidrive

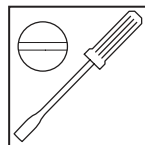

Screwdriver (Flat)

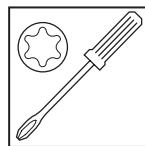

Screwdriver (Torx)

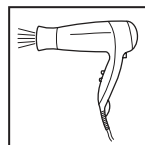




Föhn

## Recommended torque

| Thread size | Pitch (mm) | Max. torque (Nm) |
|-------------|------------|------------------|
| M4          | 0.7        | 3                |
| M5          | 0.8        | 6                |
| M6          | 1          | 10               |
| M8          | 1.25       | 25               |



The minimum torque values are 7-9% below the maximum values

## INNHold

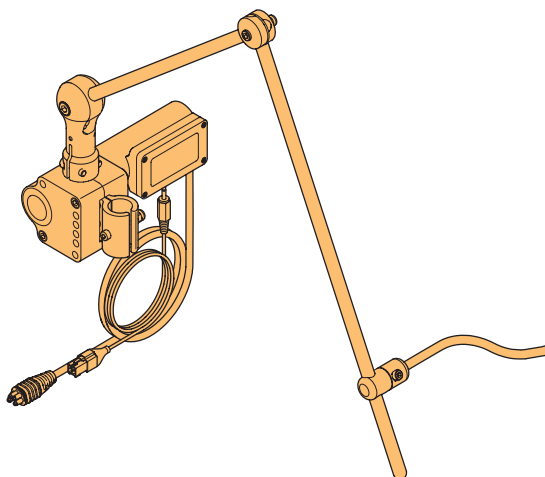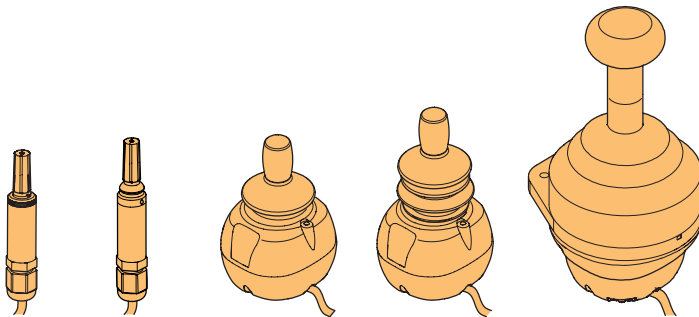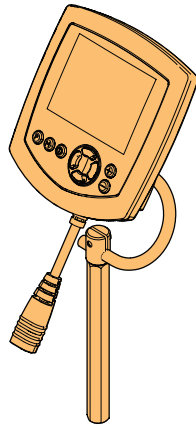





## 1.1 Omni





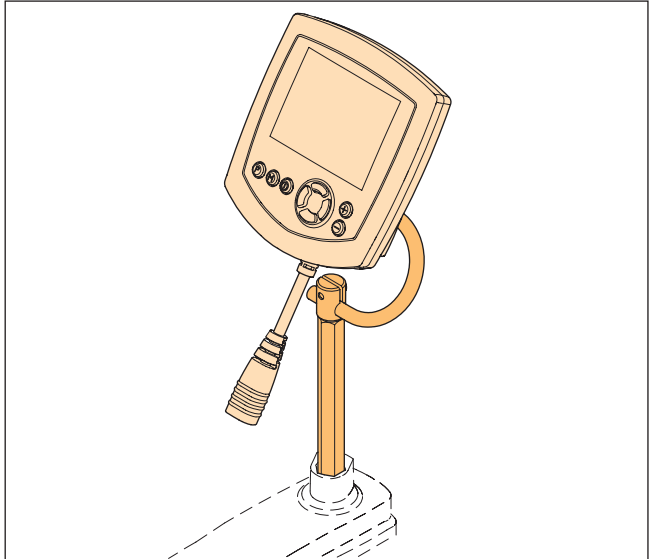
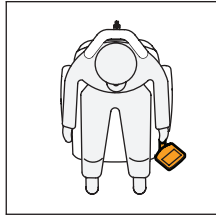


## 1.1.1 Omni (parts)

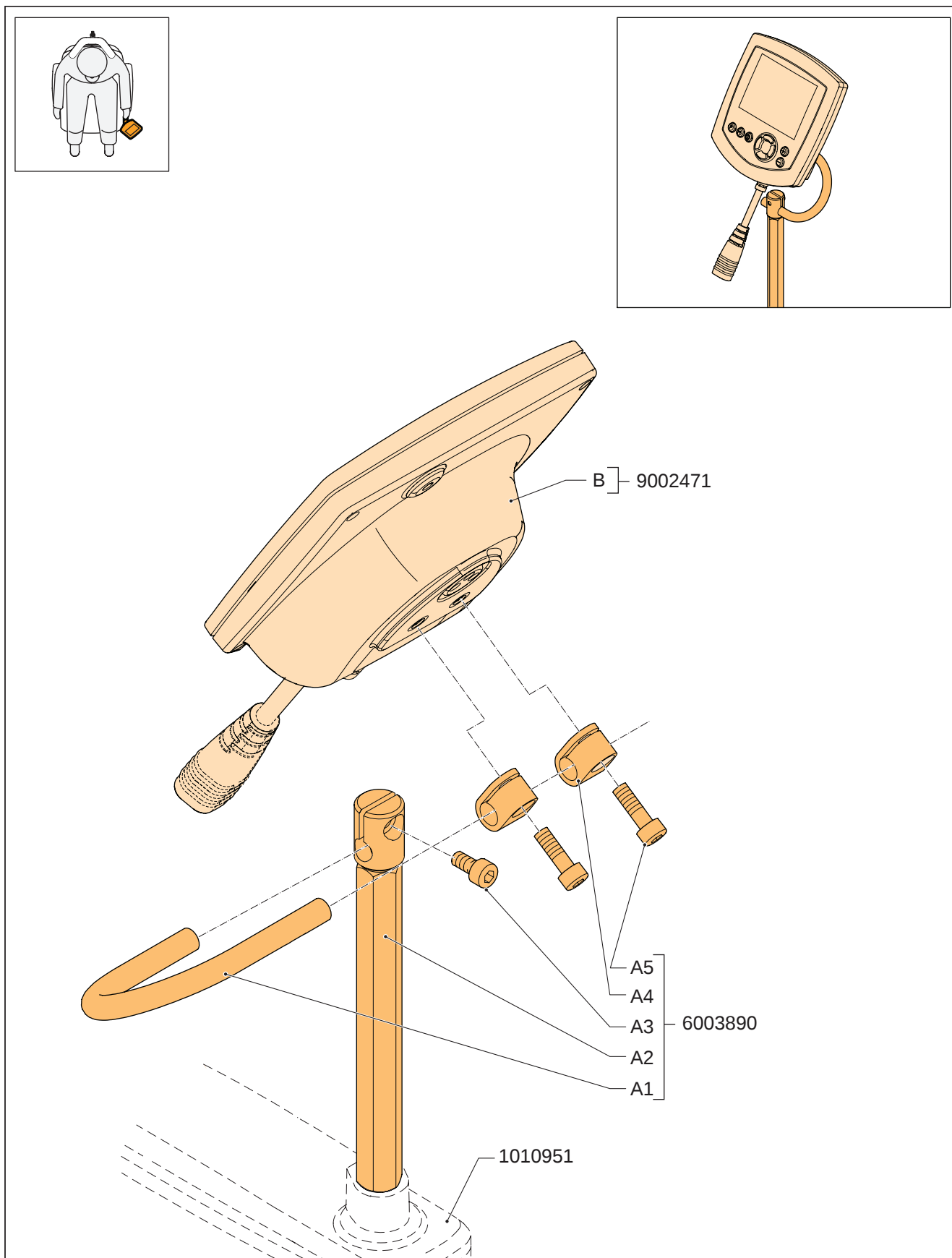






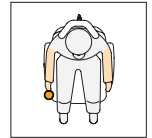


## 1.1.2 Omni (assembly)

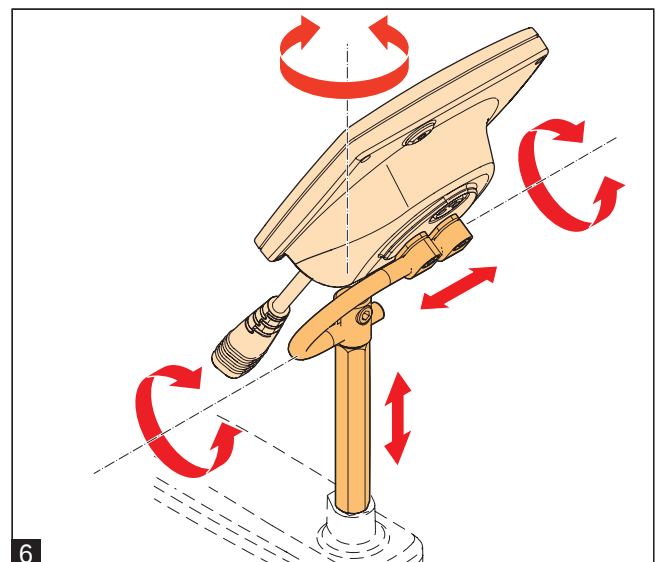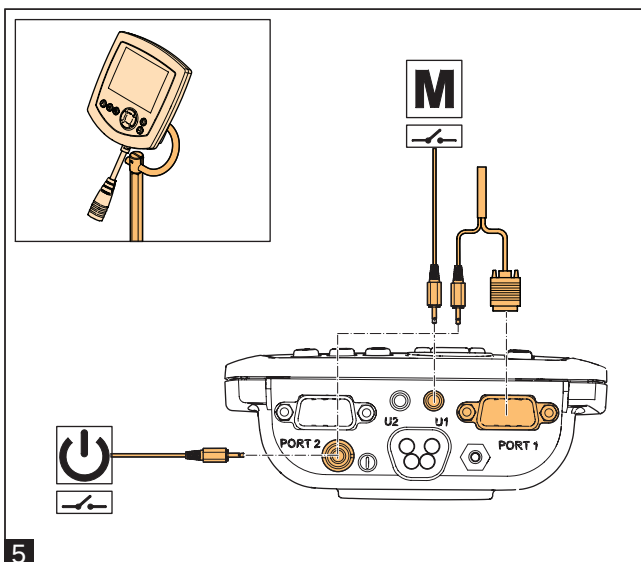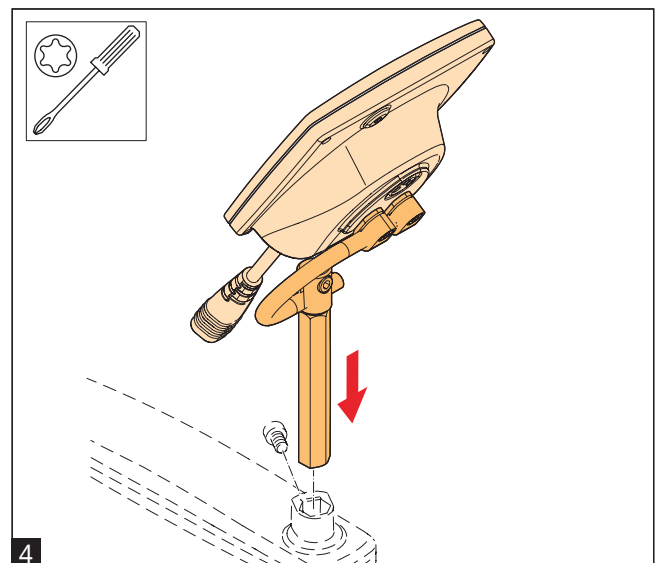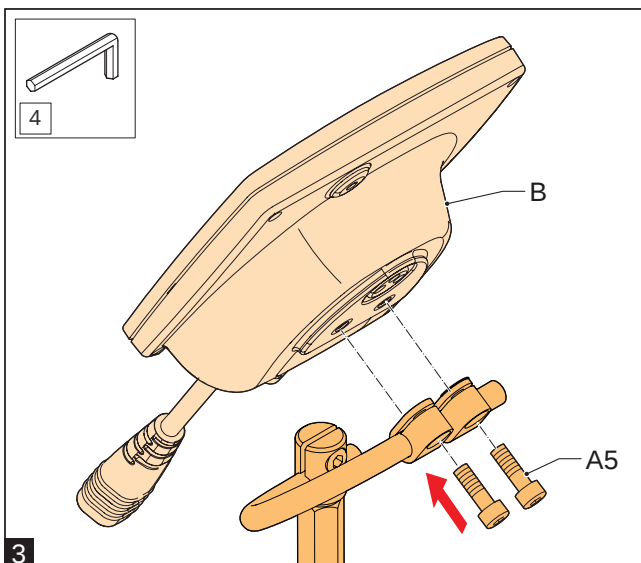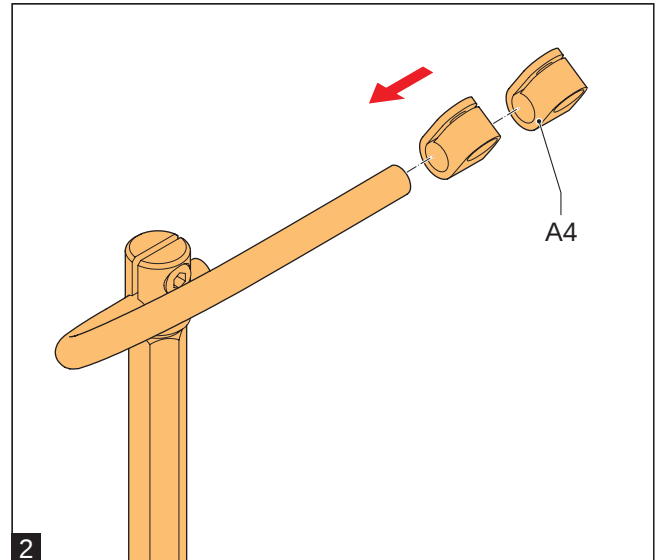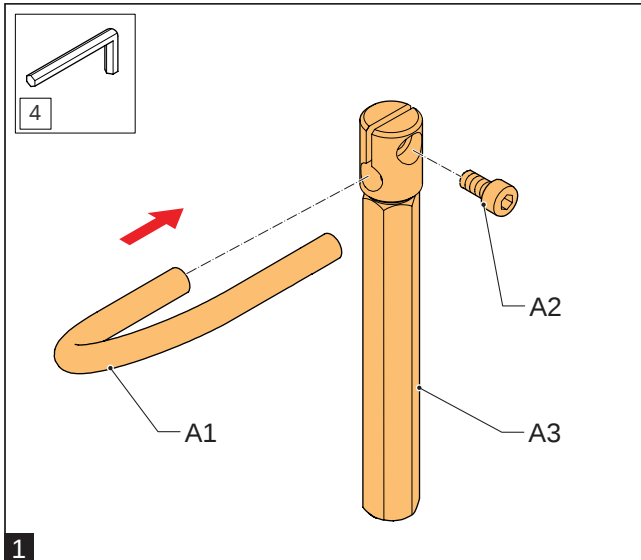






## 2.1 Micro-pilot / Micro-guide Joystick


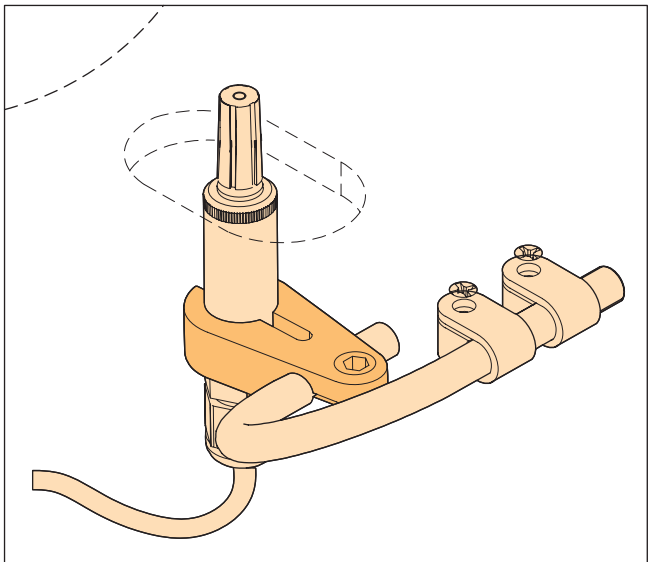
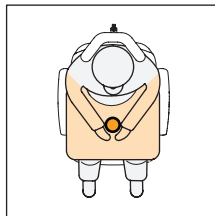
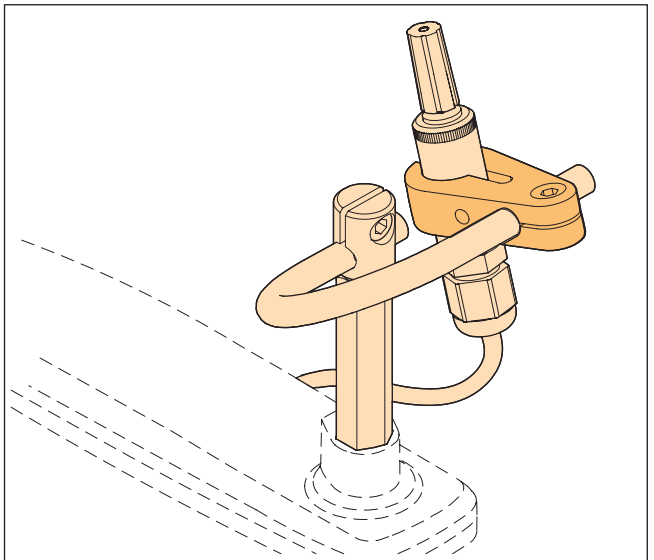
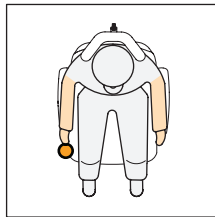
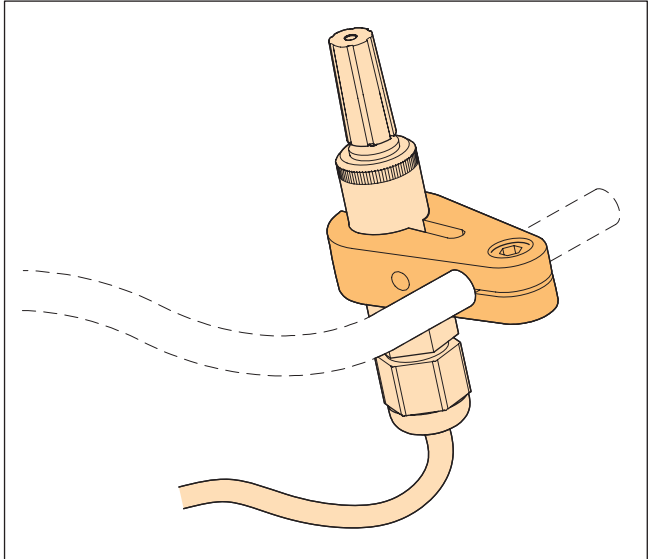
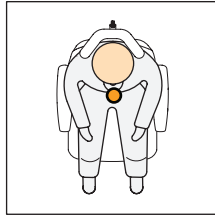


## 2.1.1 Micro-pilot / Micro-guide Joystick (parts)







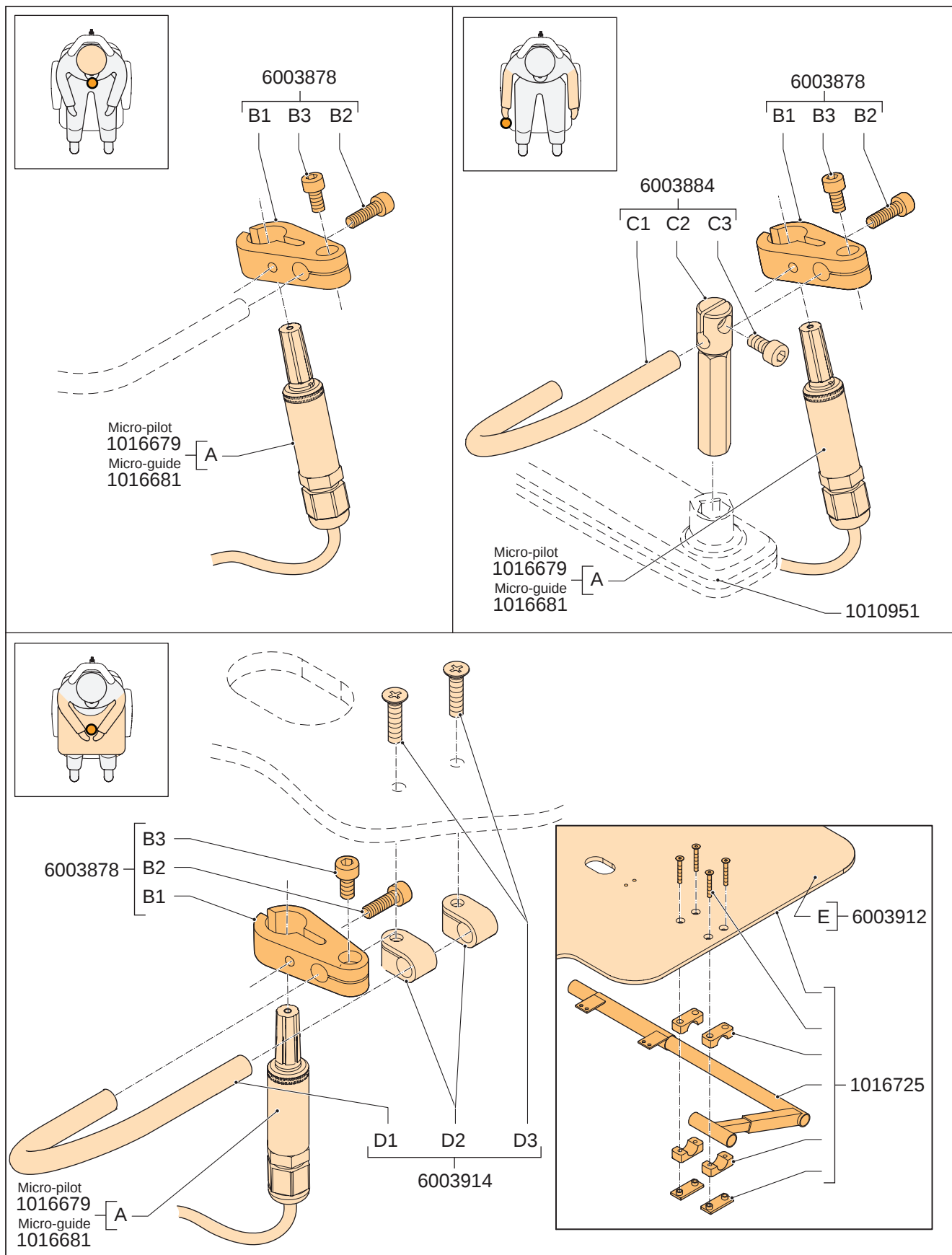


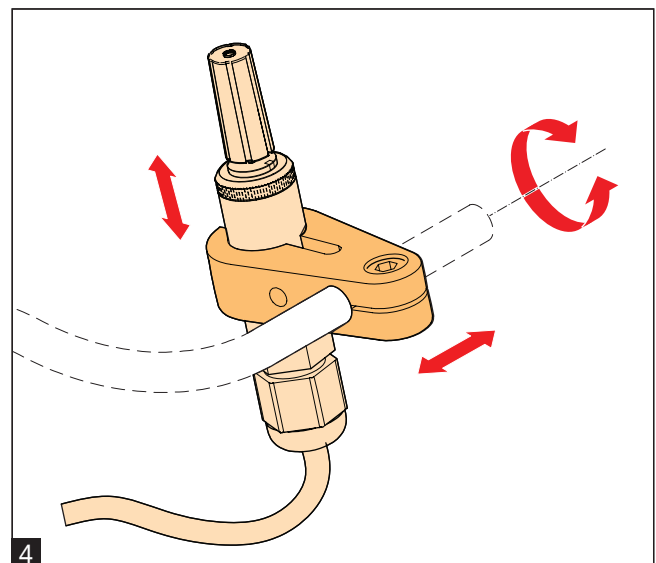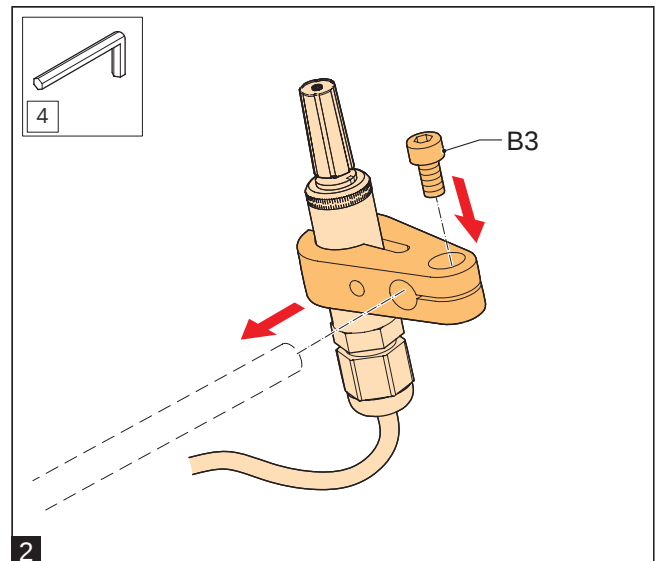

### 2.1.3 Micro-pilot / Micro-guide Joystick (assembly 2)



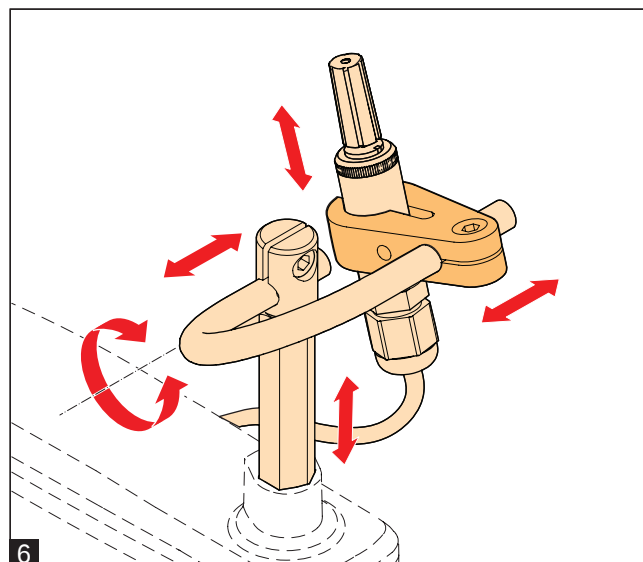
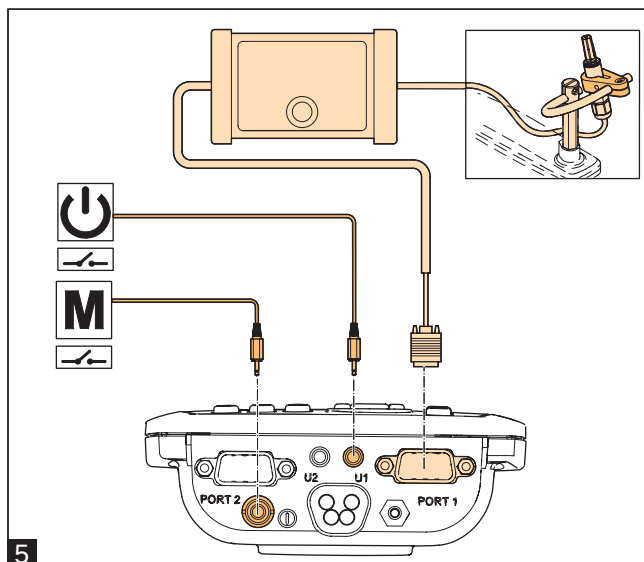
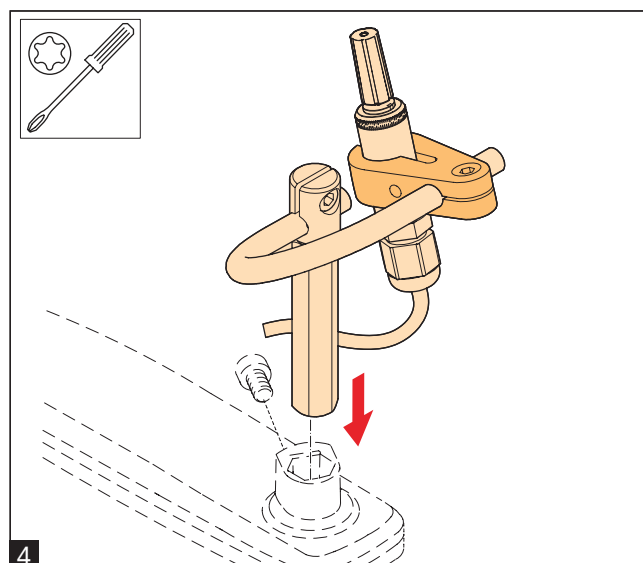
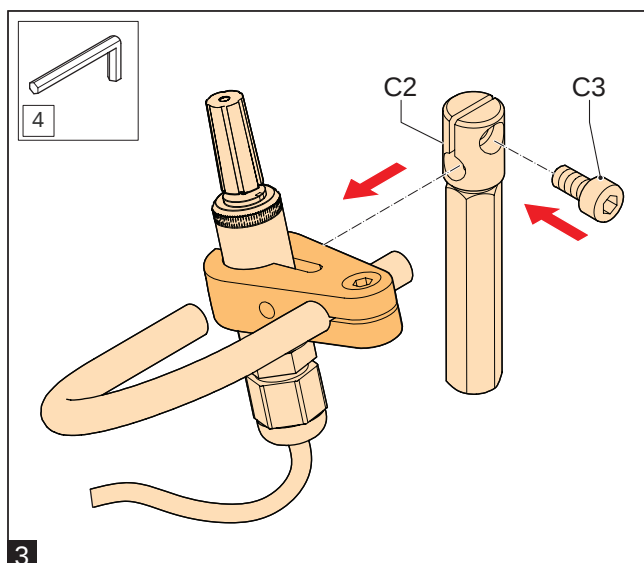
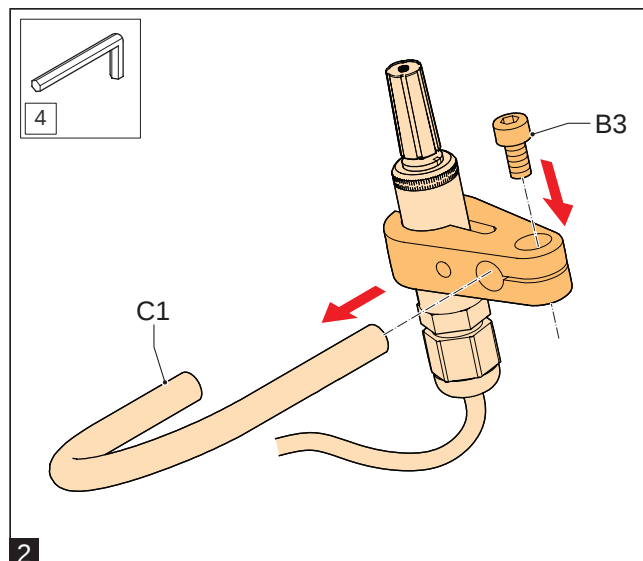
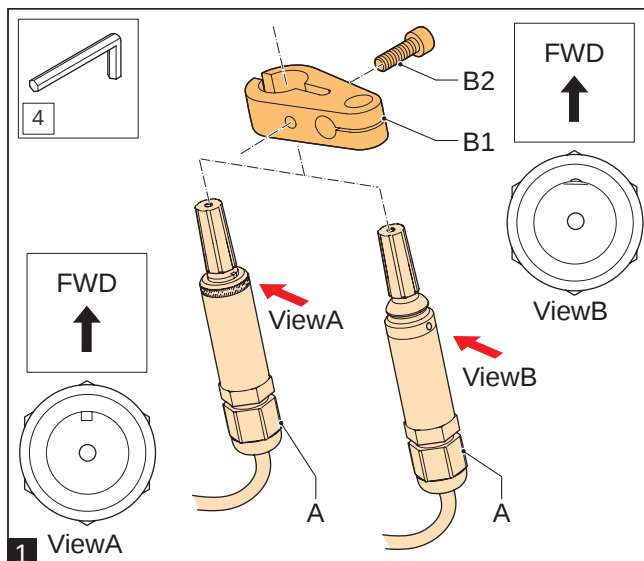
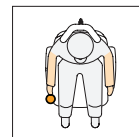


## 2.1.4 Micro-pilot / Micro-guide Joystick (assembly 3)

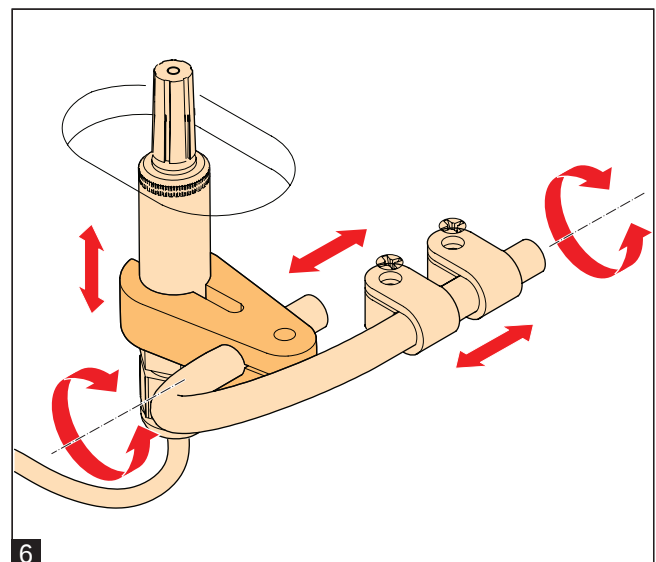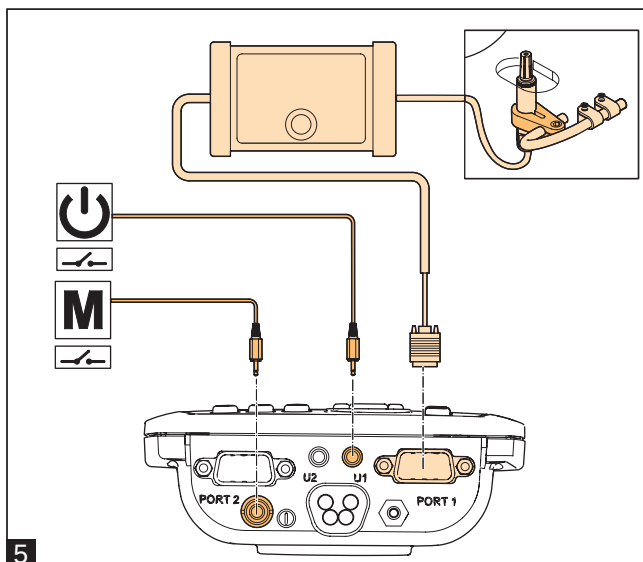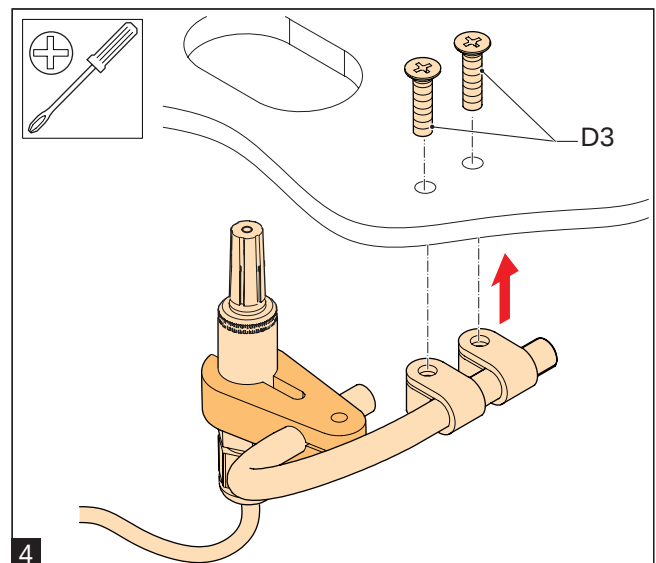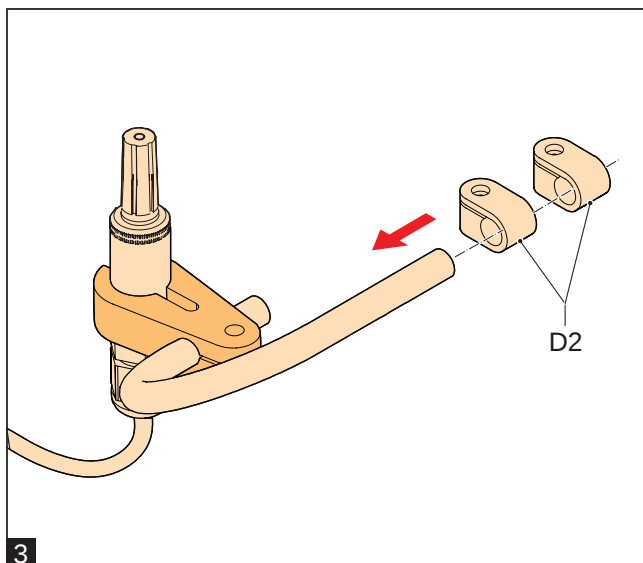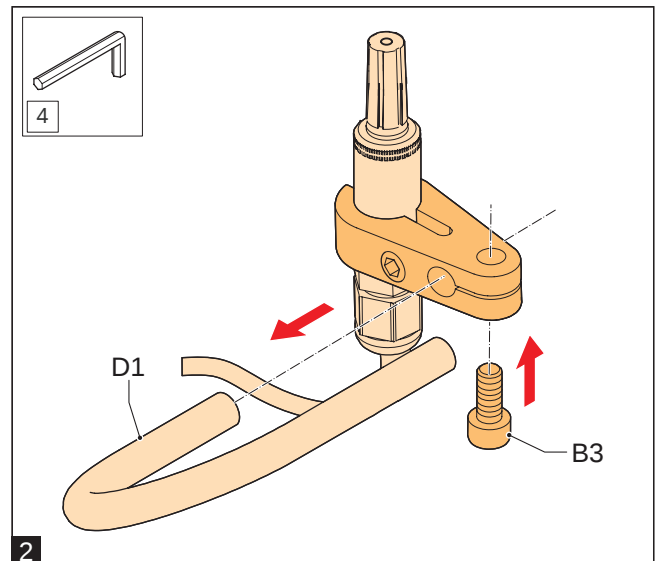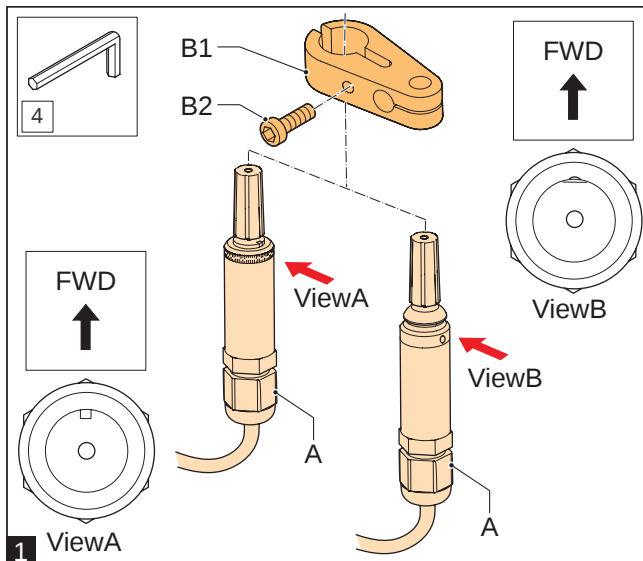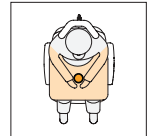





## 2.2 All-round Joystick

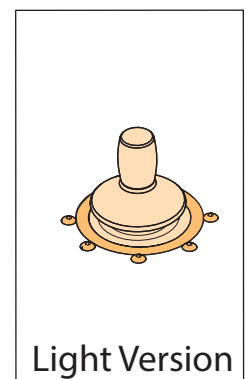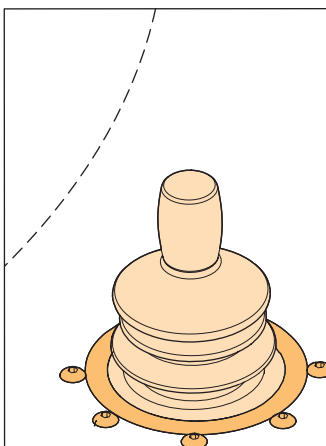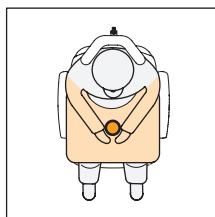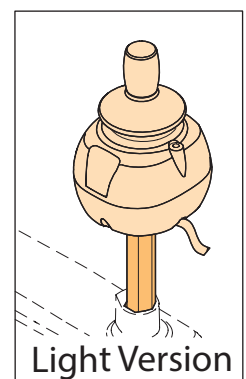
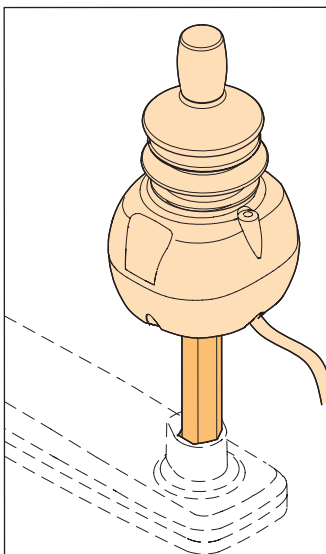
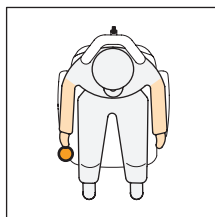
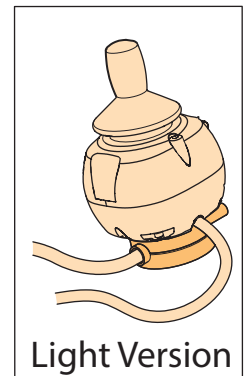
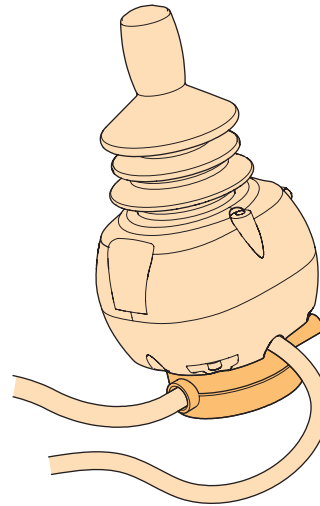
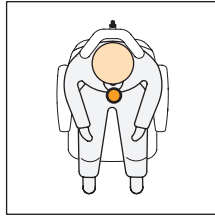



## 2.2.1 All-round Joystick (parts)

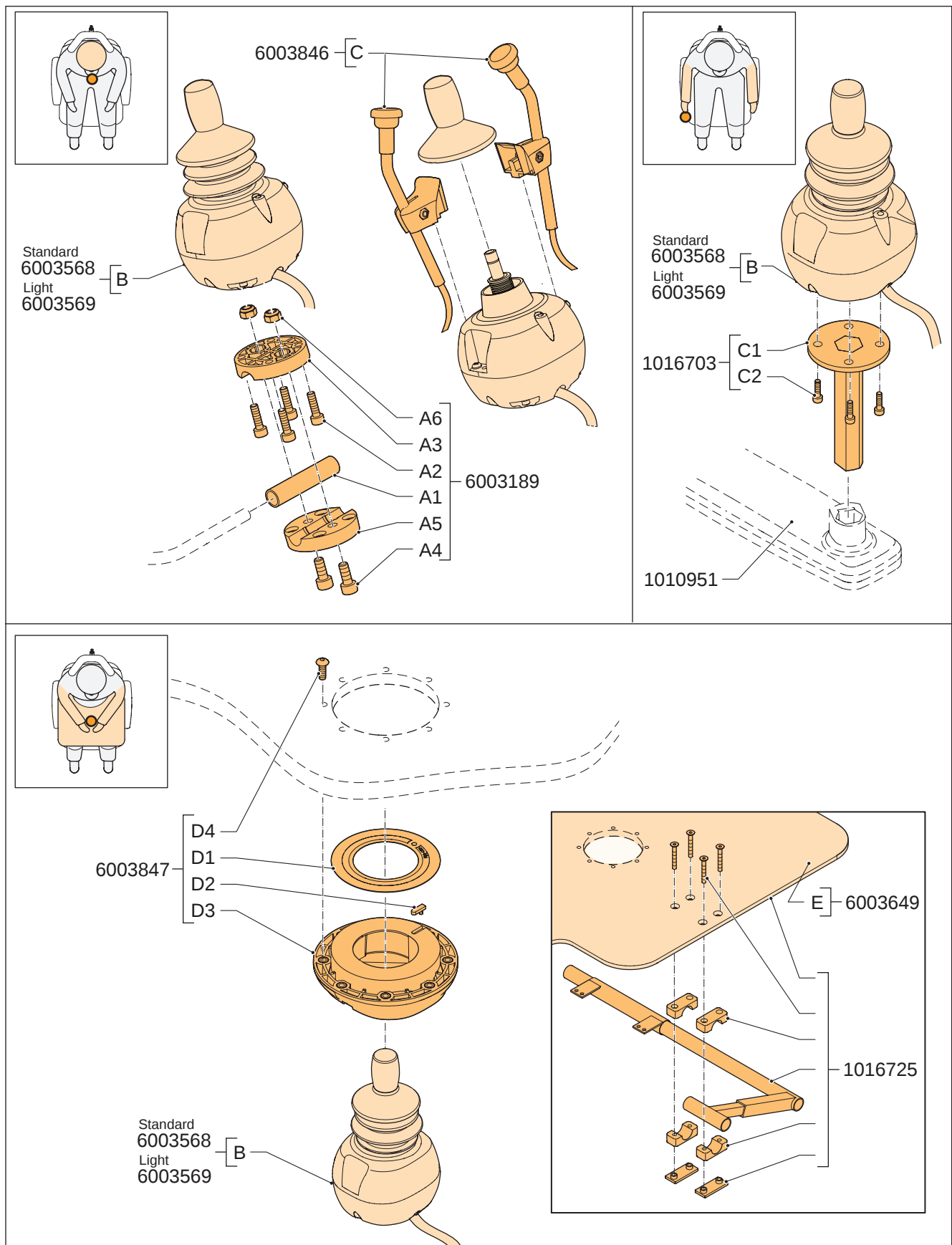




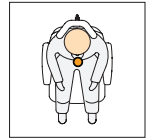


## 2.2.2 All-round Joystick (assembly 1)


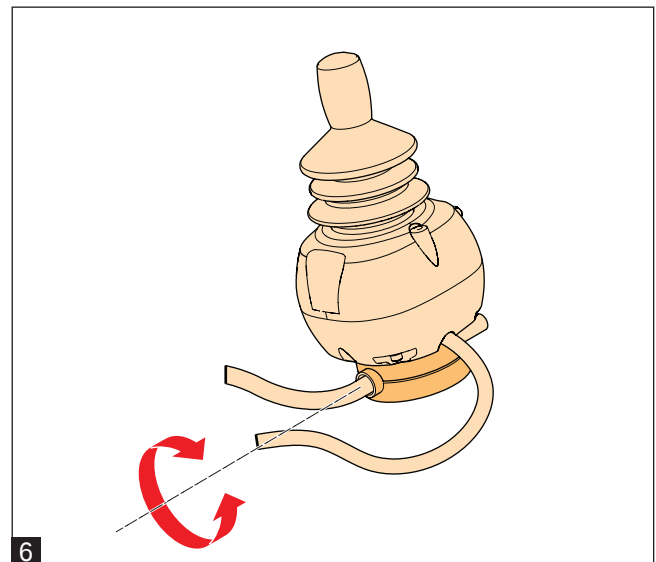
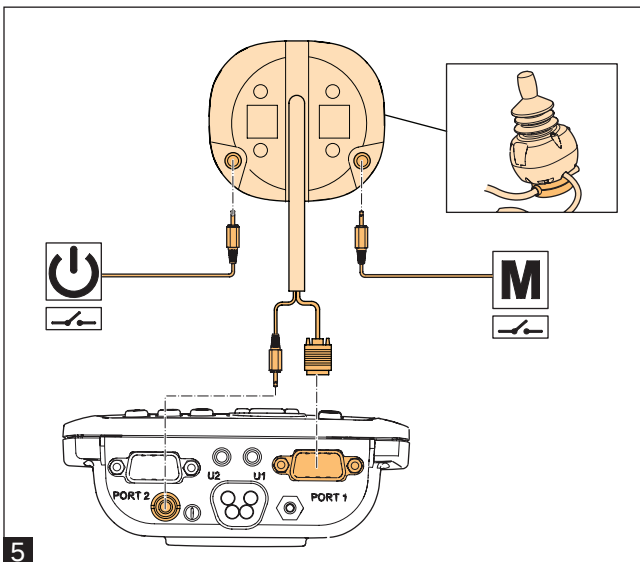
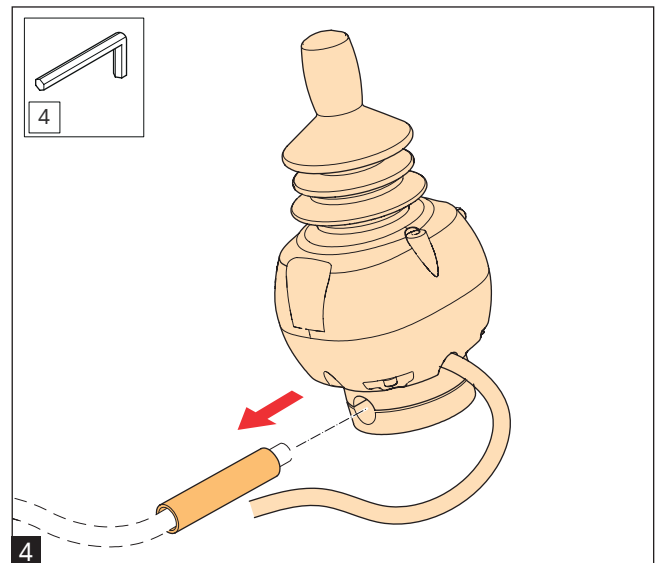
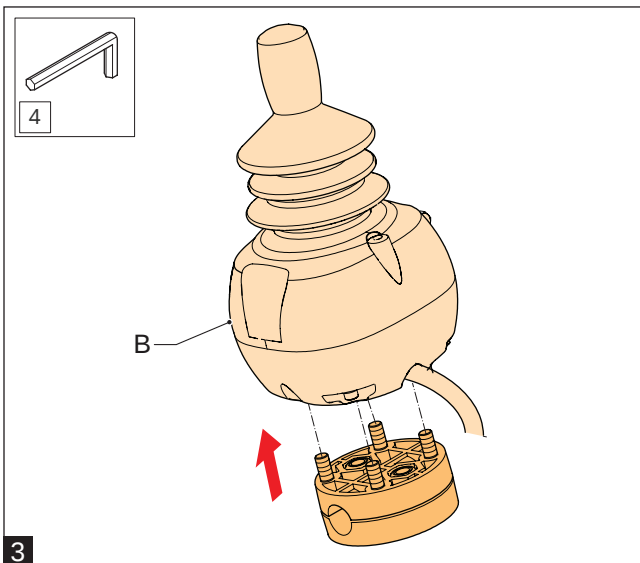
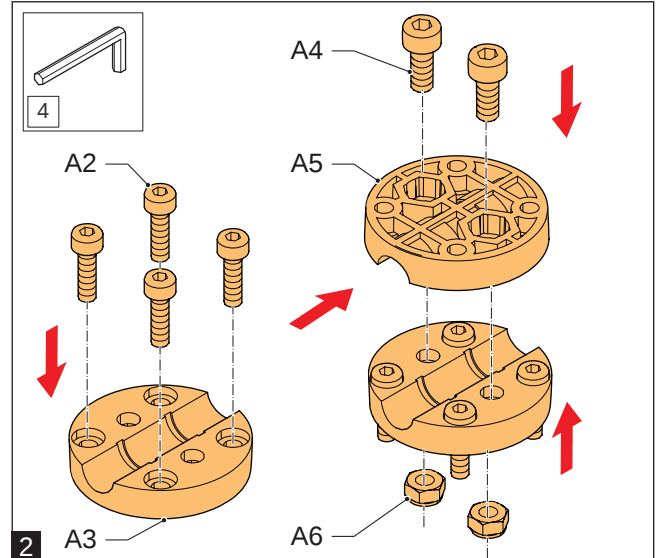
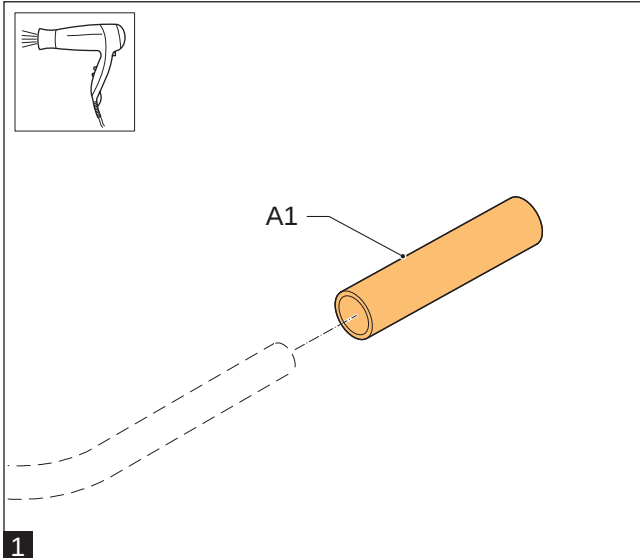


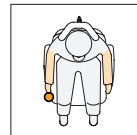



## 2.2.3 All-round Joystick (assembly 2)

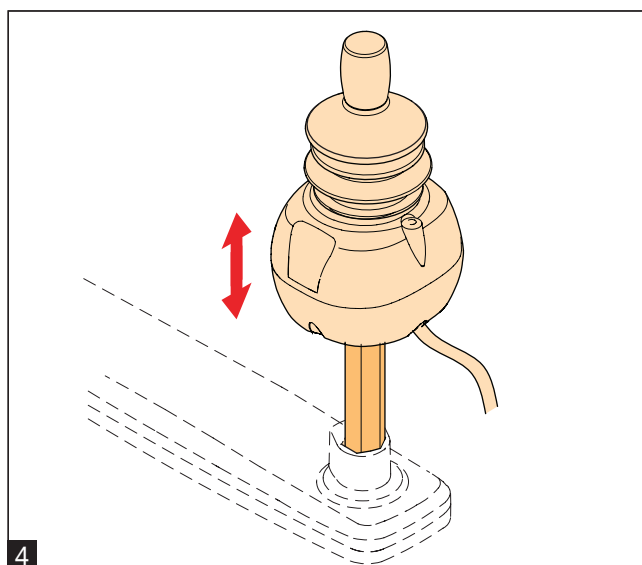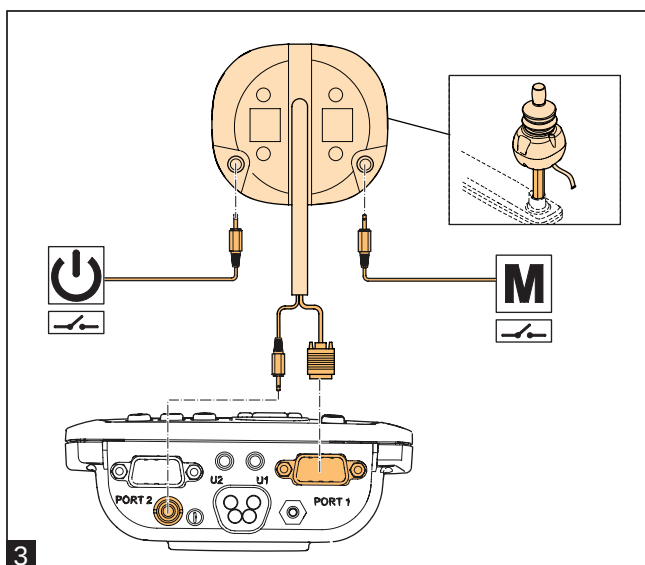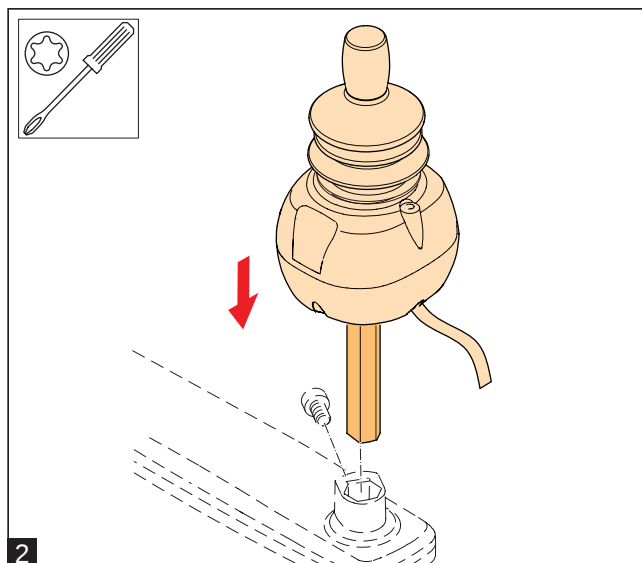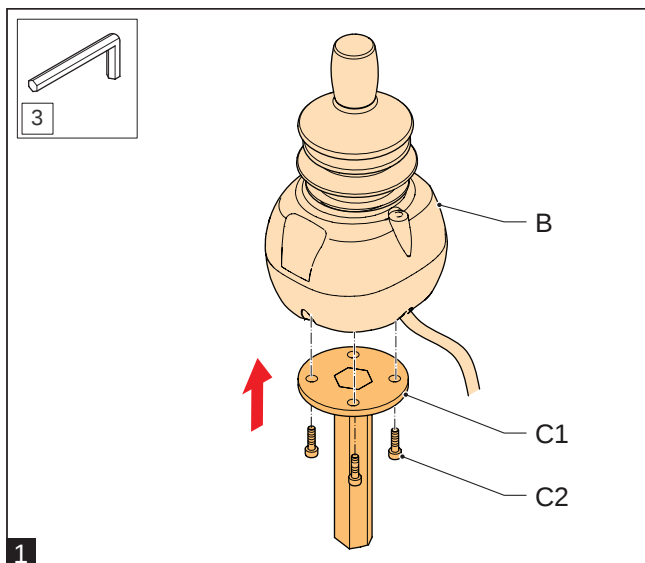

## 2.2.4 All-round Joystick (assembly 3)

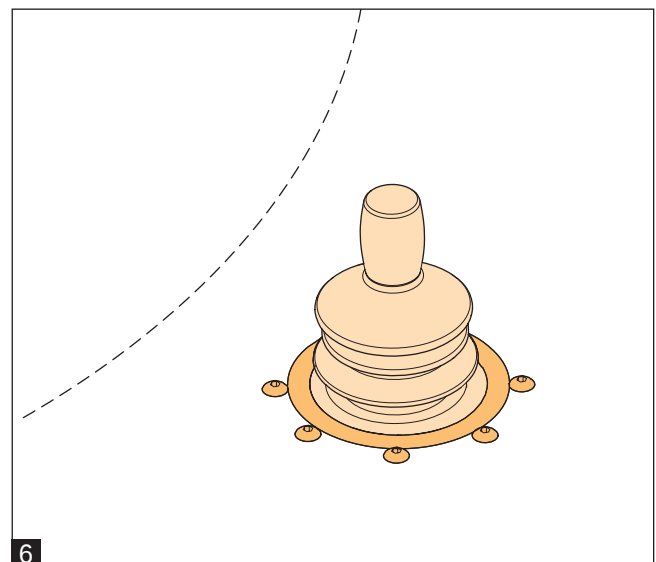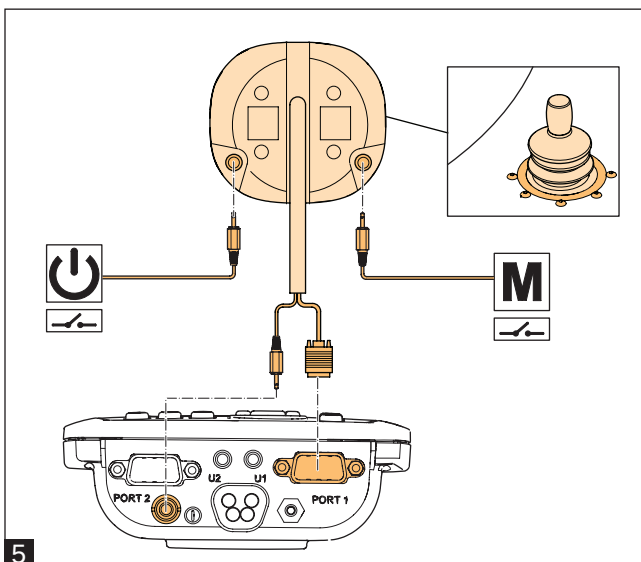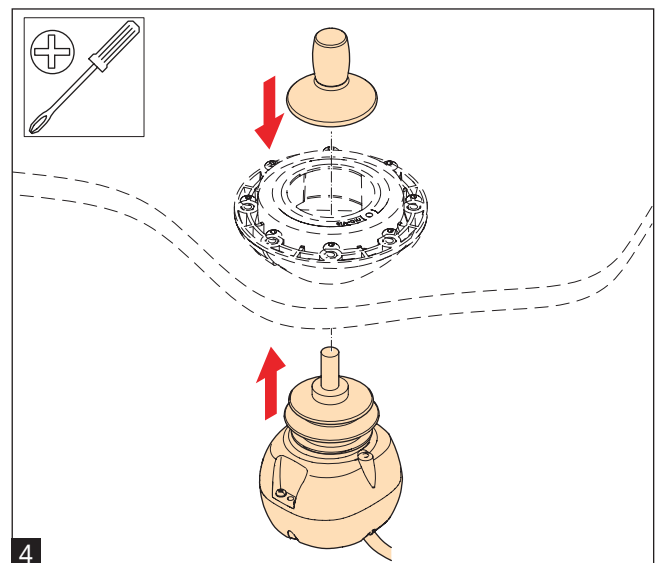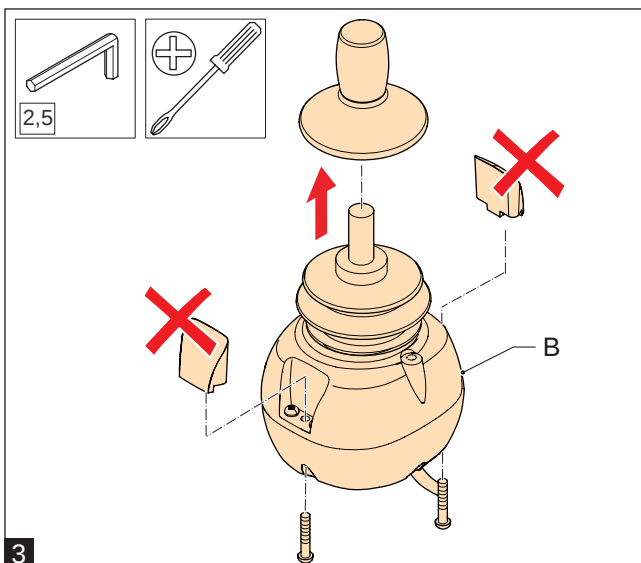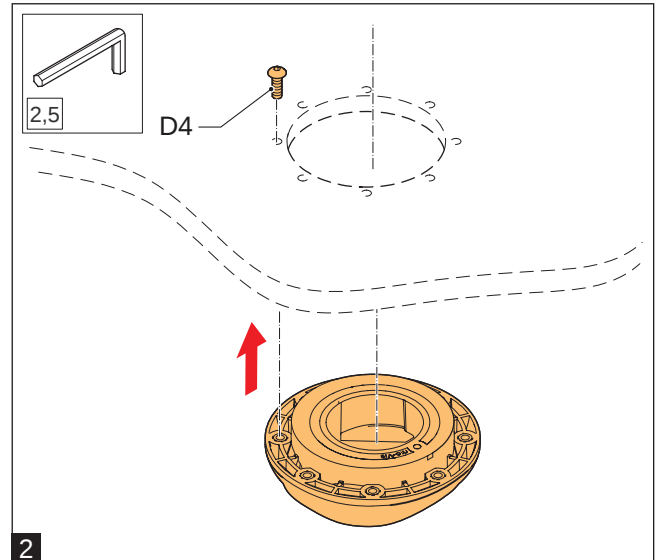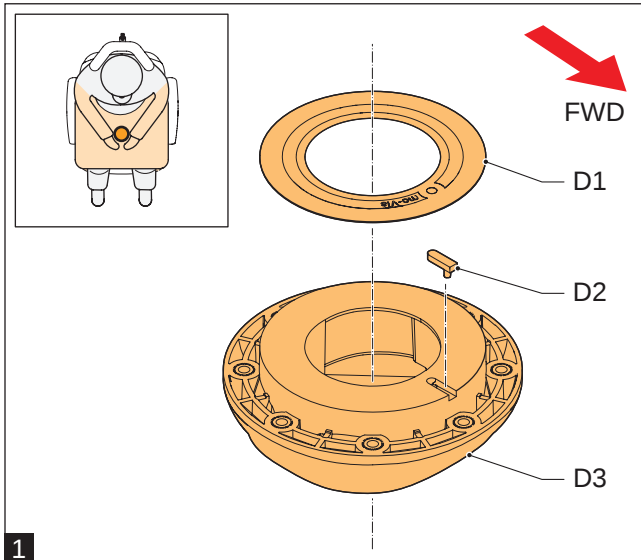
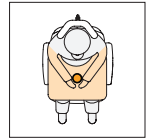


## 2.4.5 All-round Joystick (ombouw)



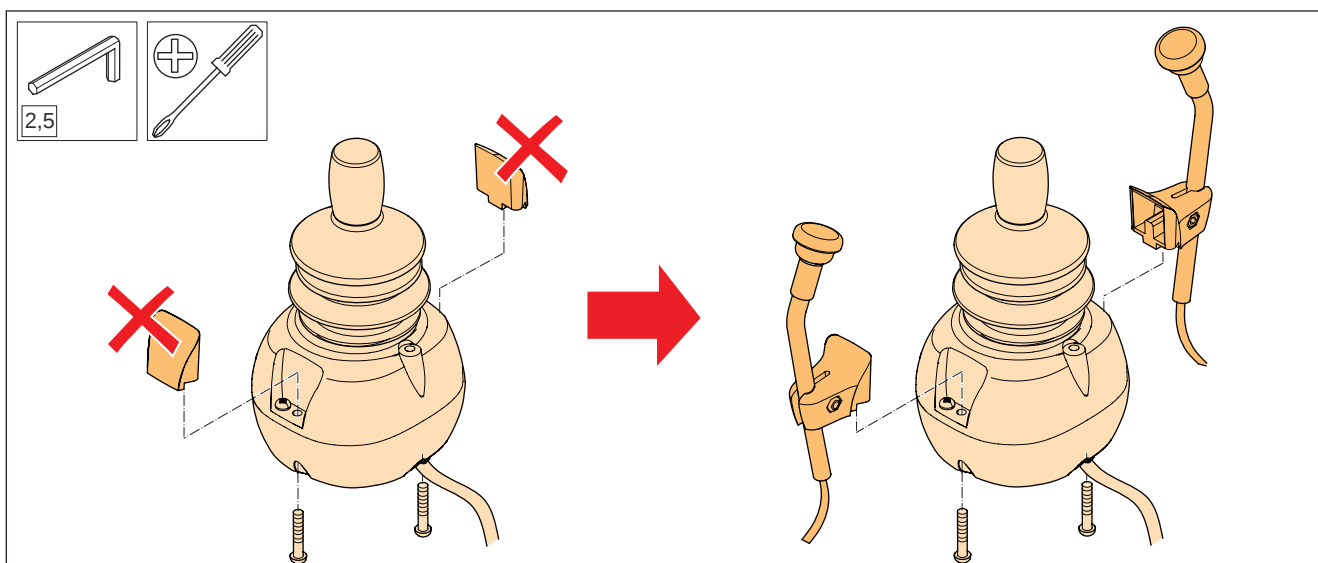
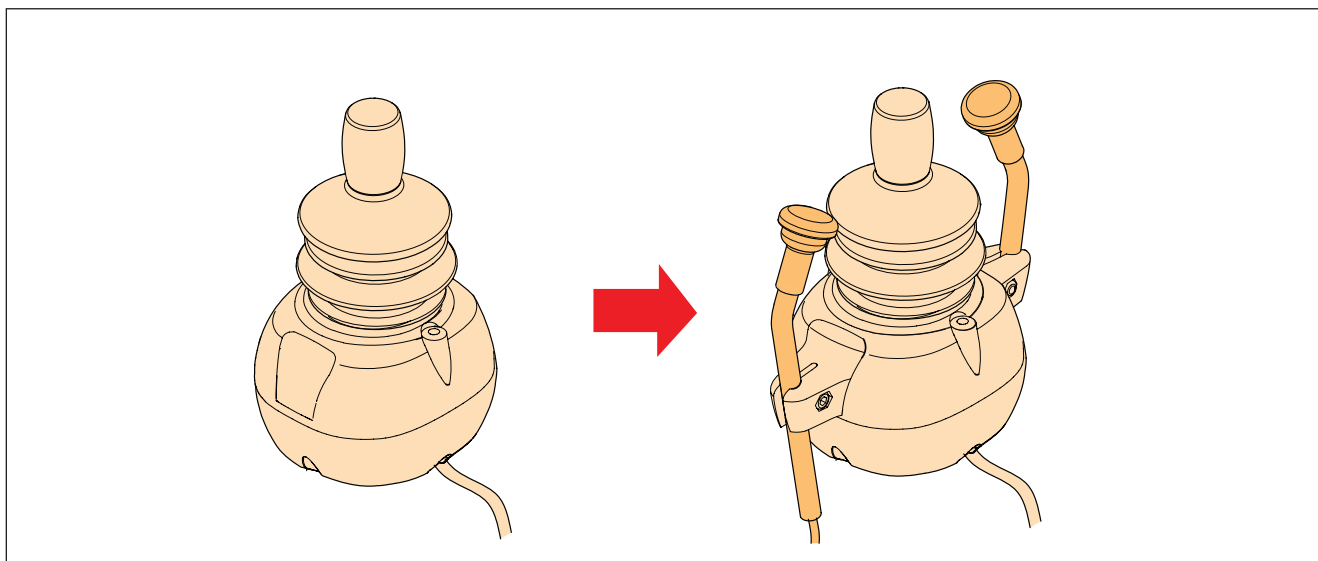


### SAFETY INSTRUCTION


Use the **All Round Joystick Light** only in combination with these two knobs!

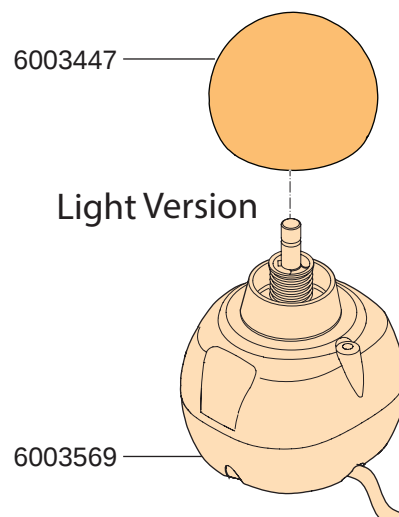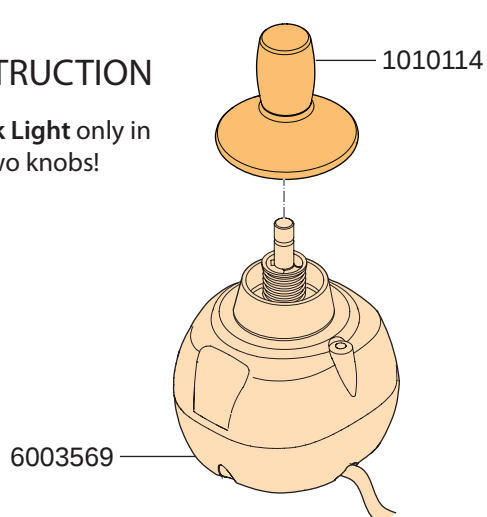


## 2.3 Heavy Duty Joystick

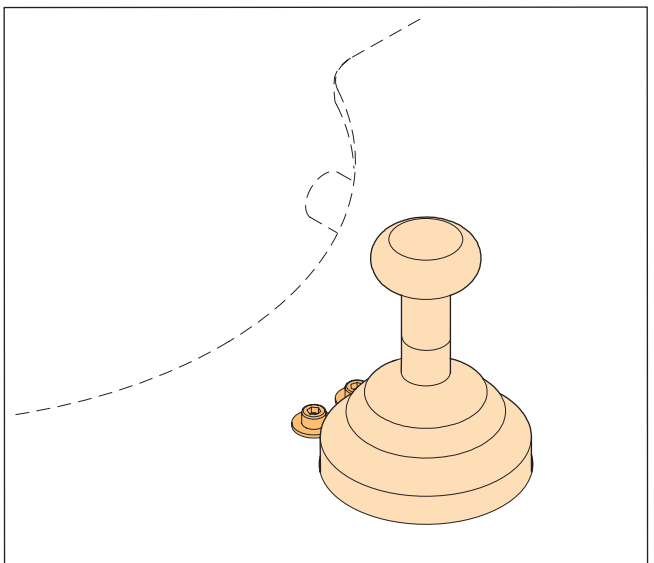
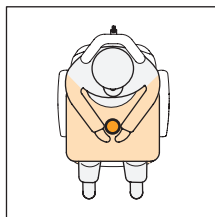
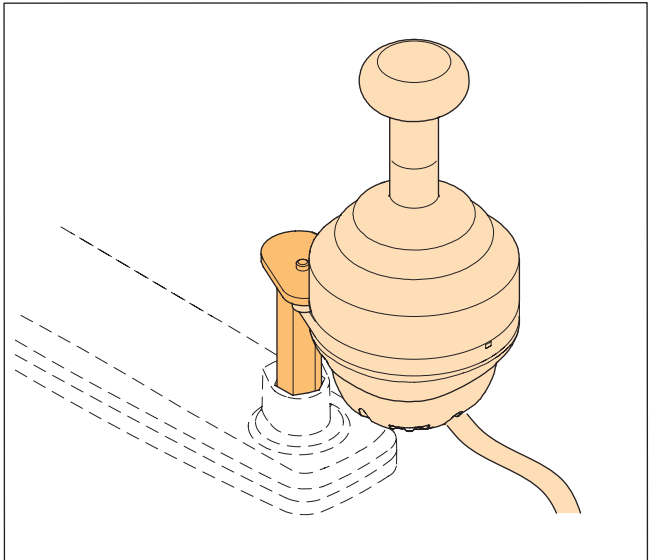
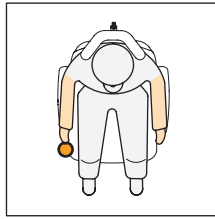


### 2.3.1 Heavy Duty Joystick (parts)




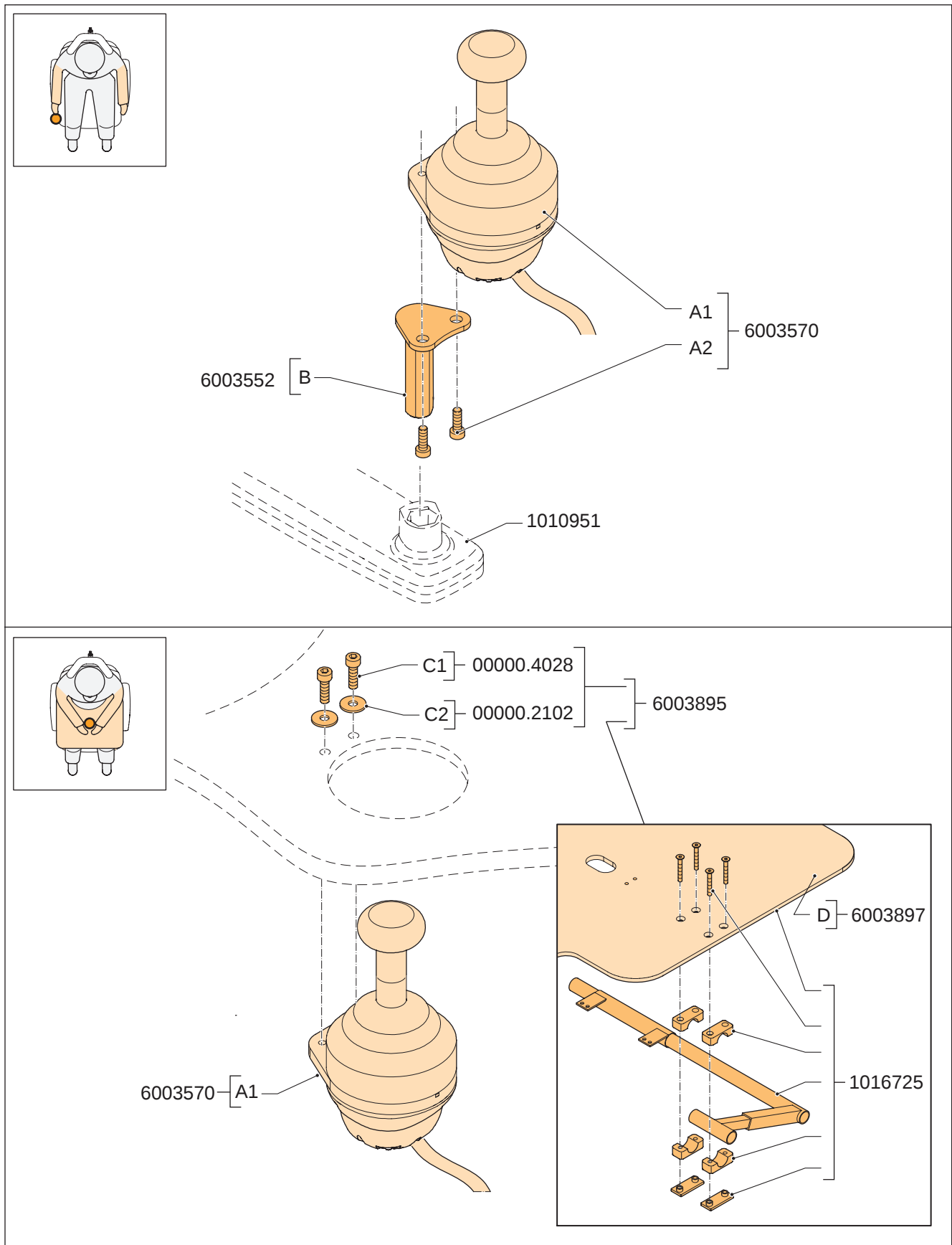


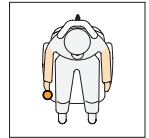

## 2.3.2 Heavy Duty Joystick (assembly 1)

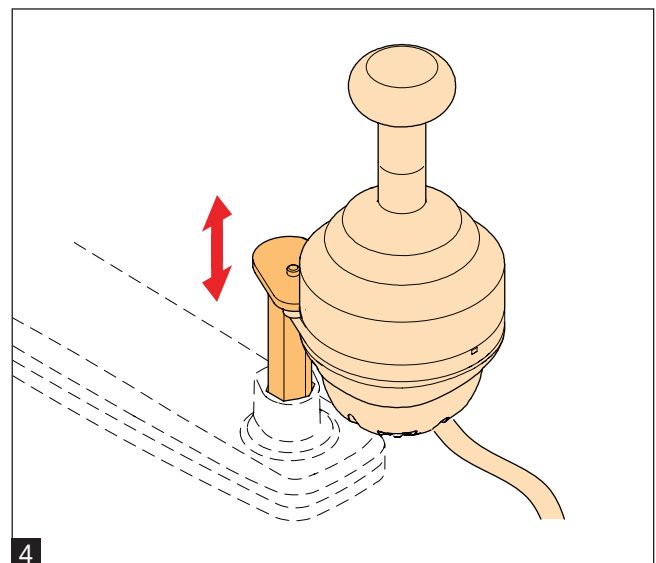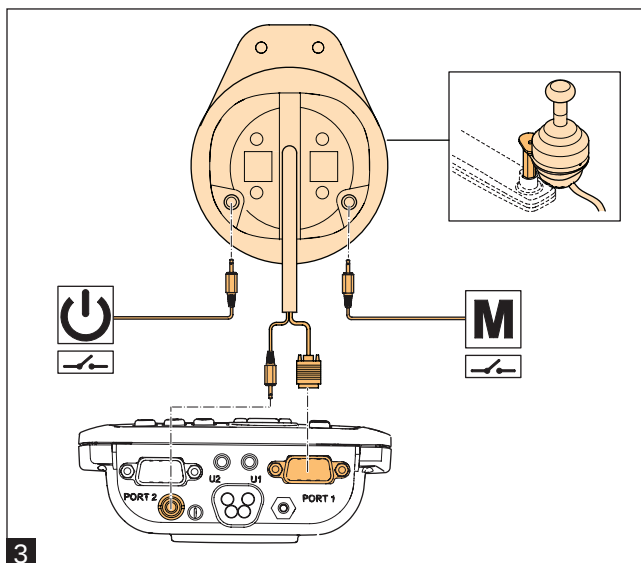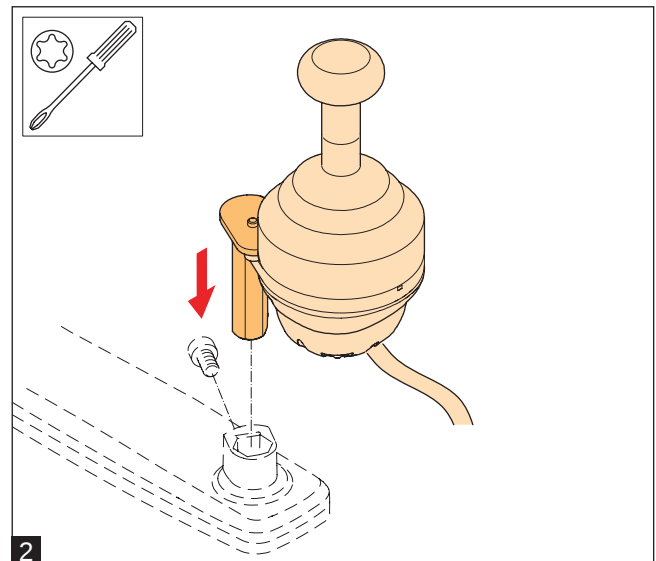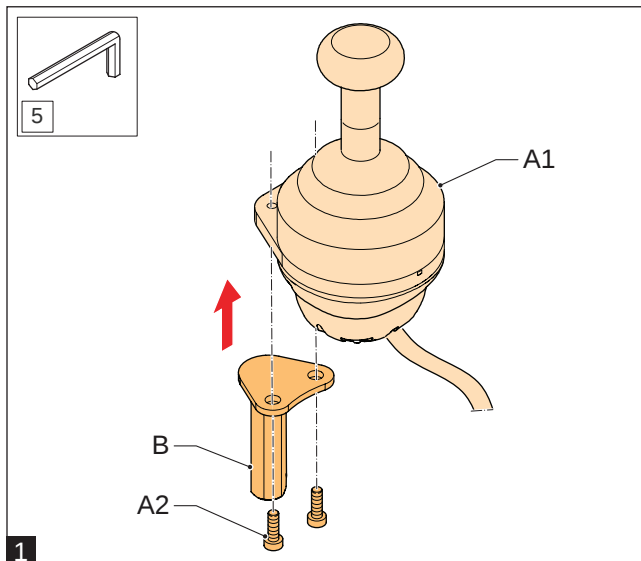


### 2.3.3 Heavy Duty Joystick (assembly 2)

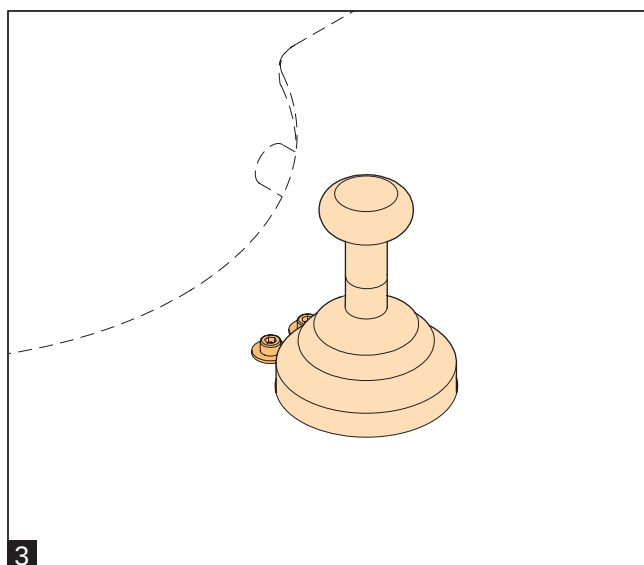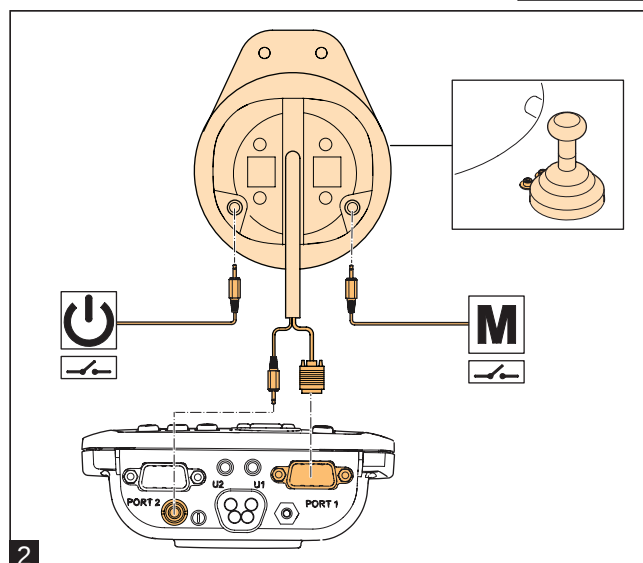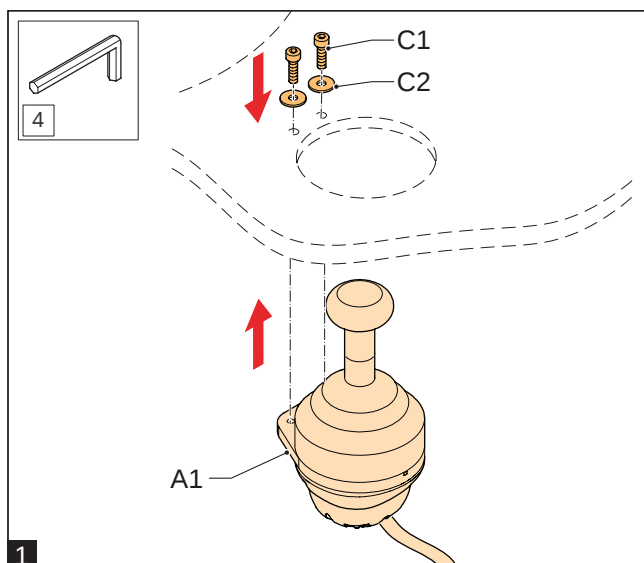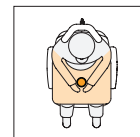


### 3.1 Chin Arm (Left / Right)


#### 3.1.1 Chin Arm Qlass

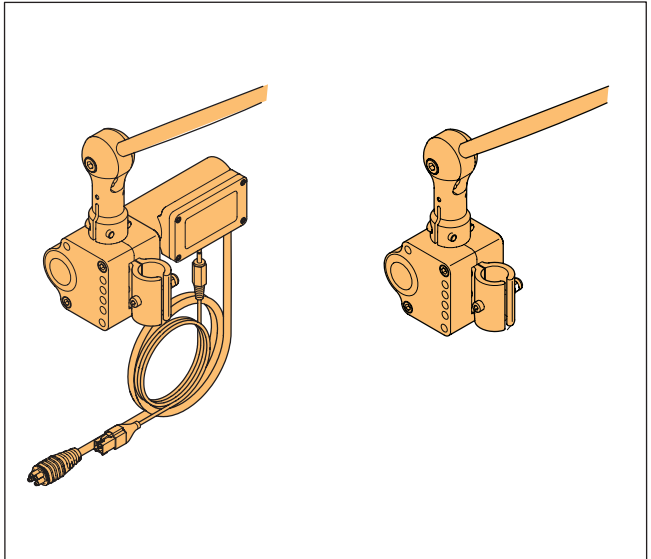

#### 3.1.2 Chin Arm Sedeo Pro

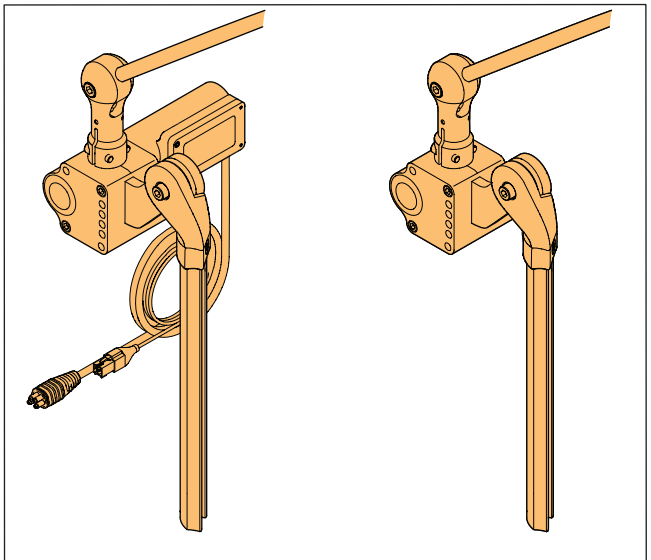


#### 3.1.3 Chin Arm Sedeo Pro+

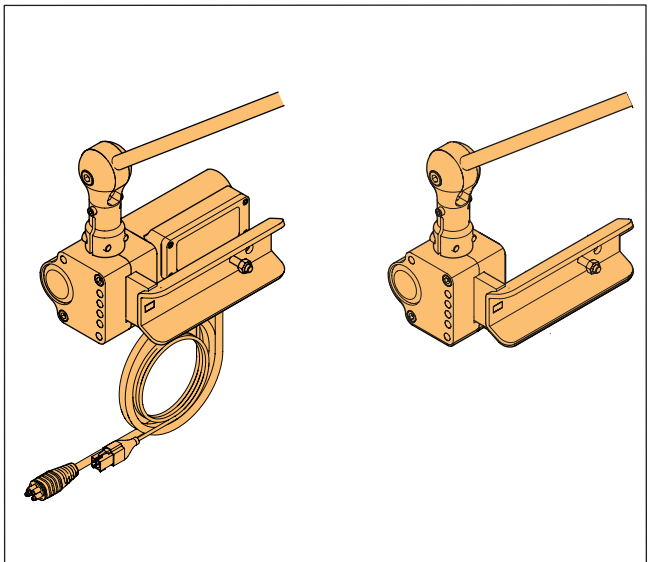


### 3.1.1 Chin Arm (Left / Right)

#### Powered Chin Arm Qlass

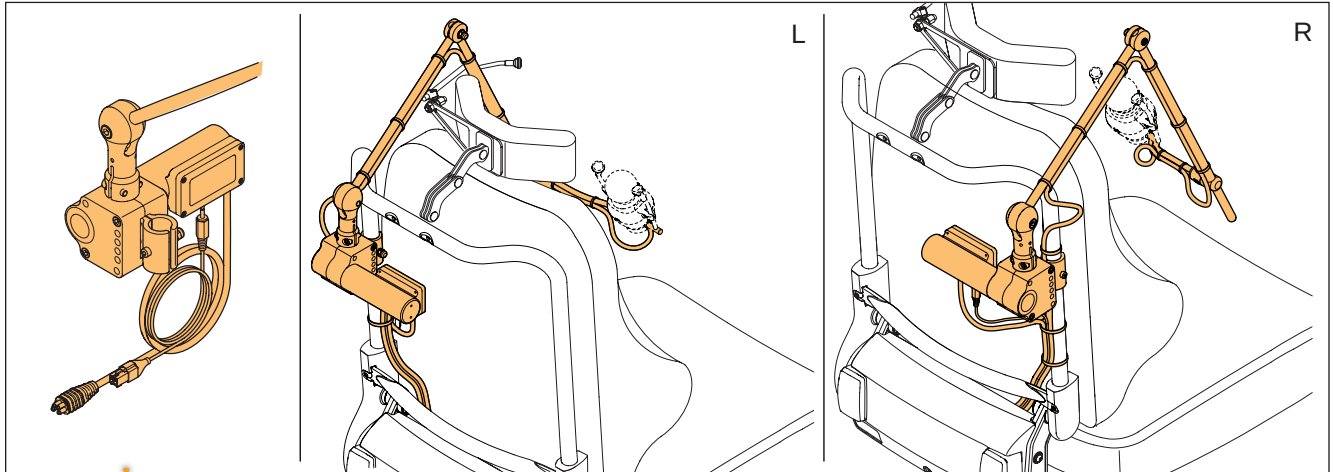


#### Manual Chin Arm Qlass

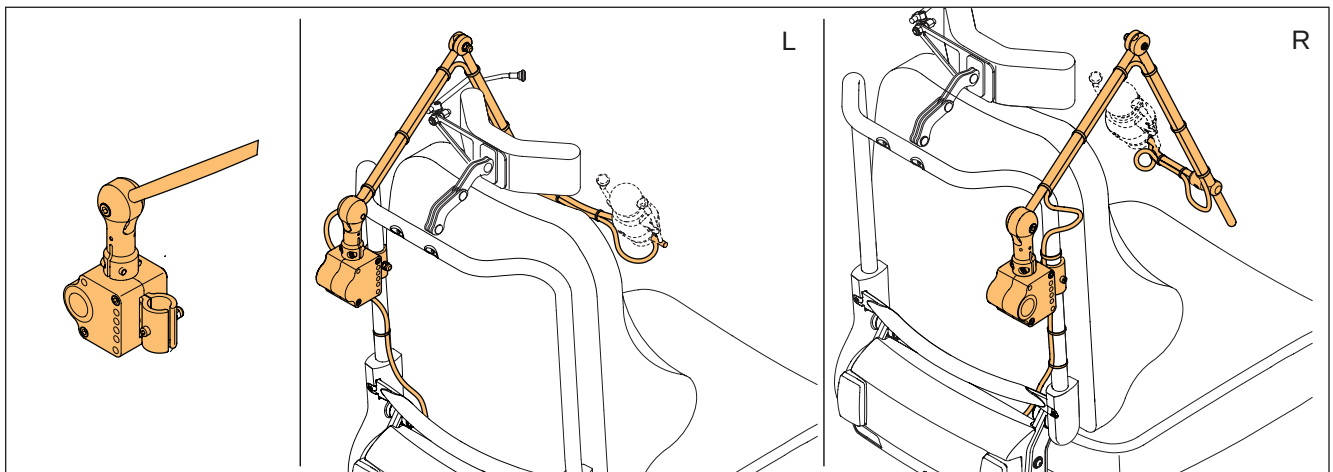


### 3.1.2 Chin Arm (Left / Right)

#### Powered Chin Arm Sedeo Pro

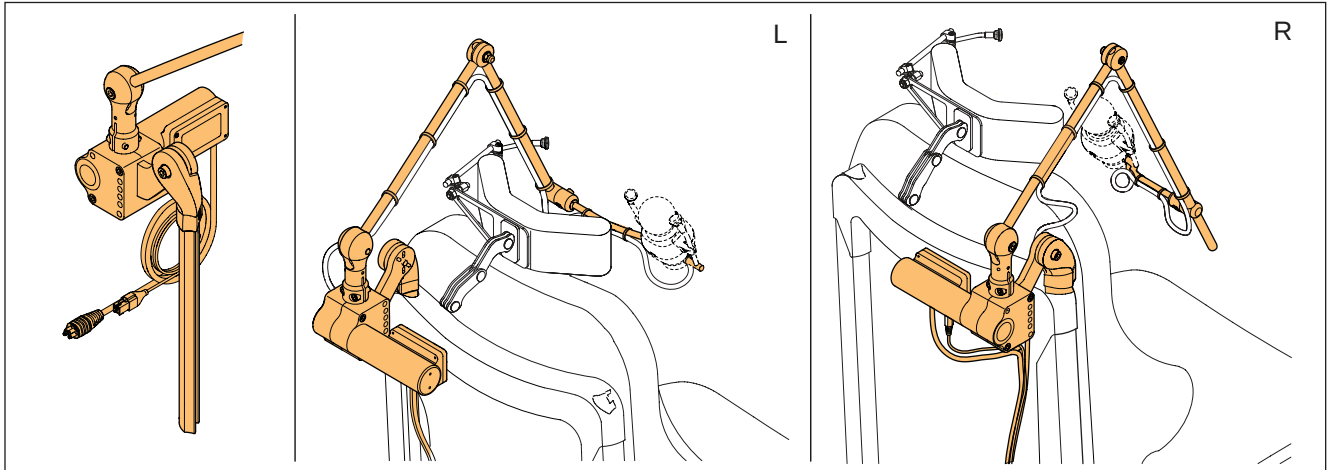


#### Manual Chin Arm Sedeo Pro

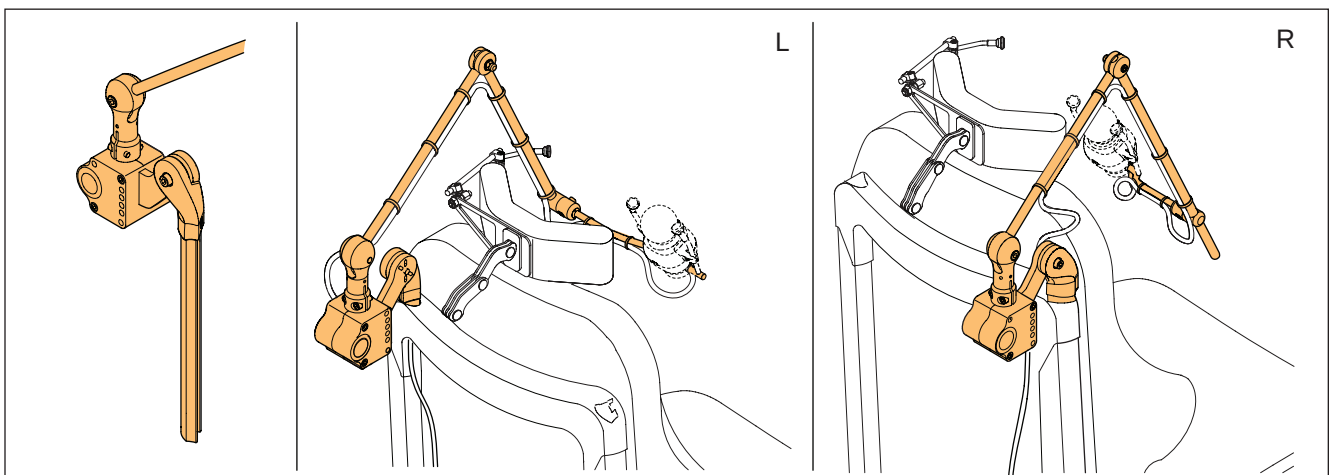


### 3.1.3 Chin Arm (Left / Right)


#### Powered Chin Arm Sedeo Pro+

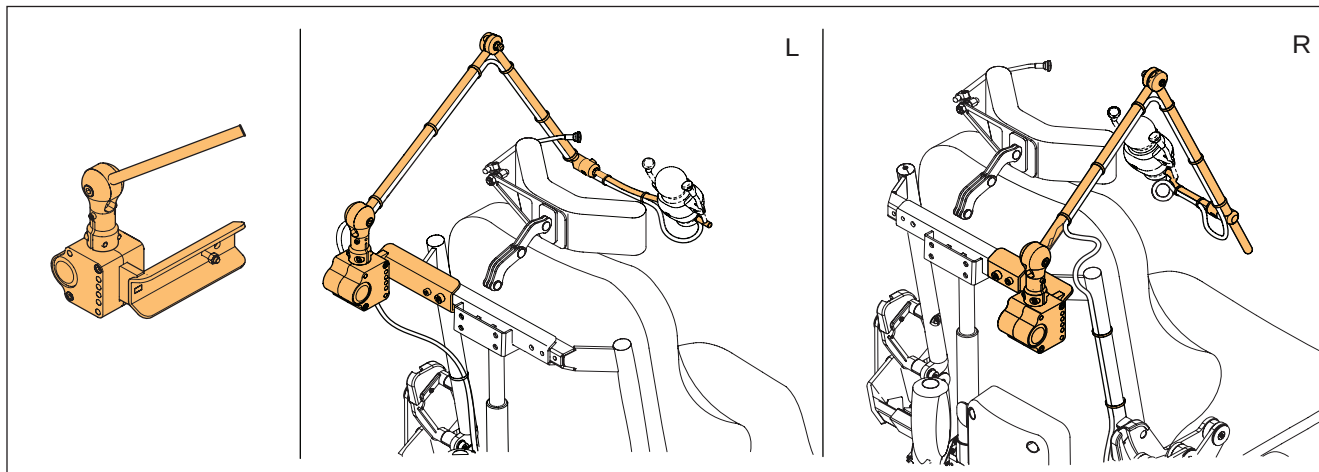


#### Manual Chin Arm Sedeo Pro+

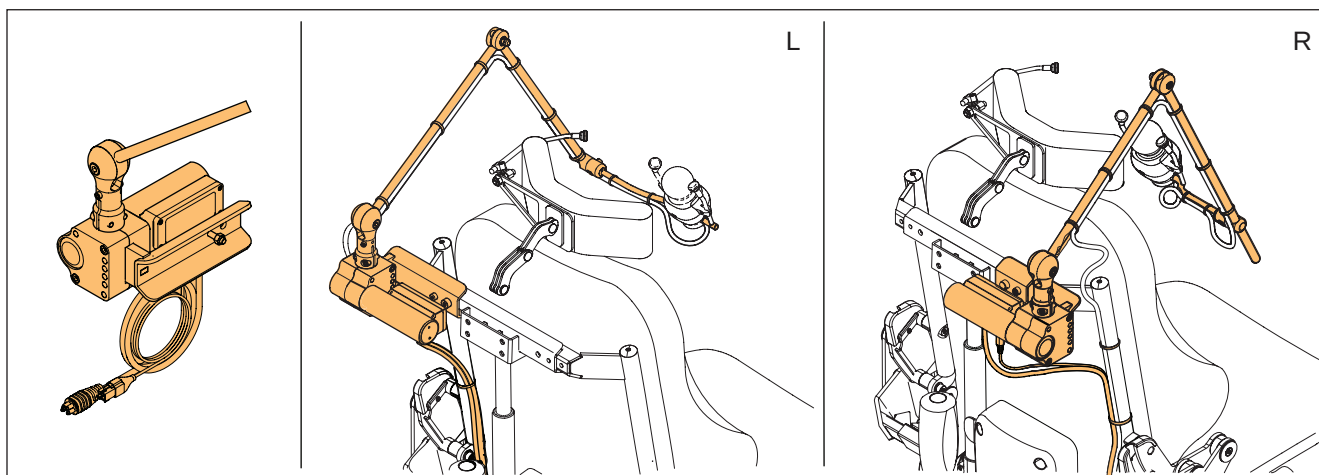







## 3.2 Chin Arm (parts)

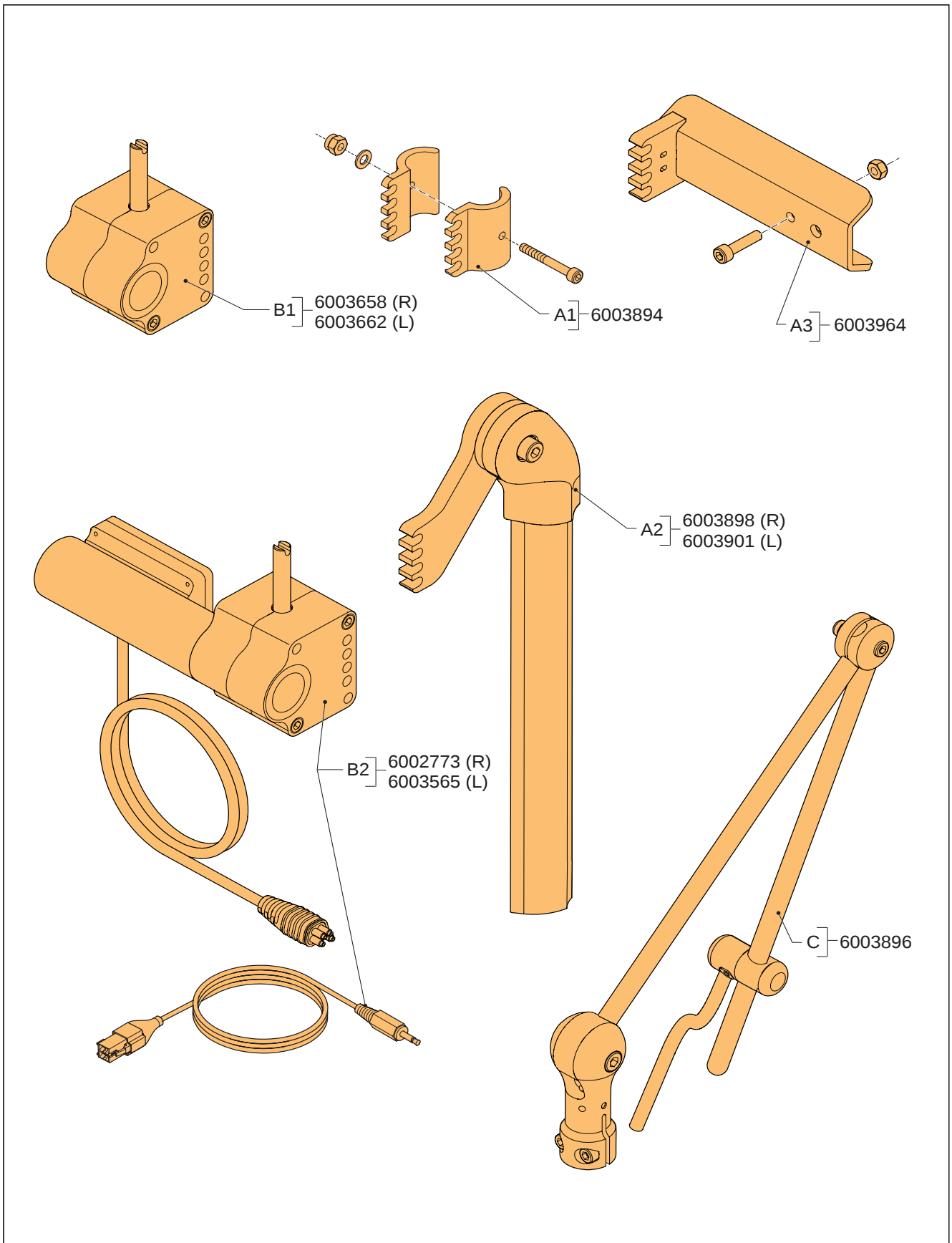







### 3.2.1 Chin Arm (assembly 1)

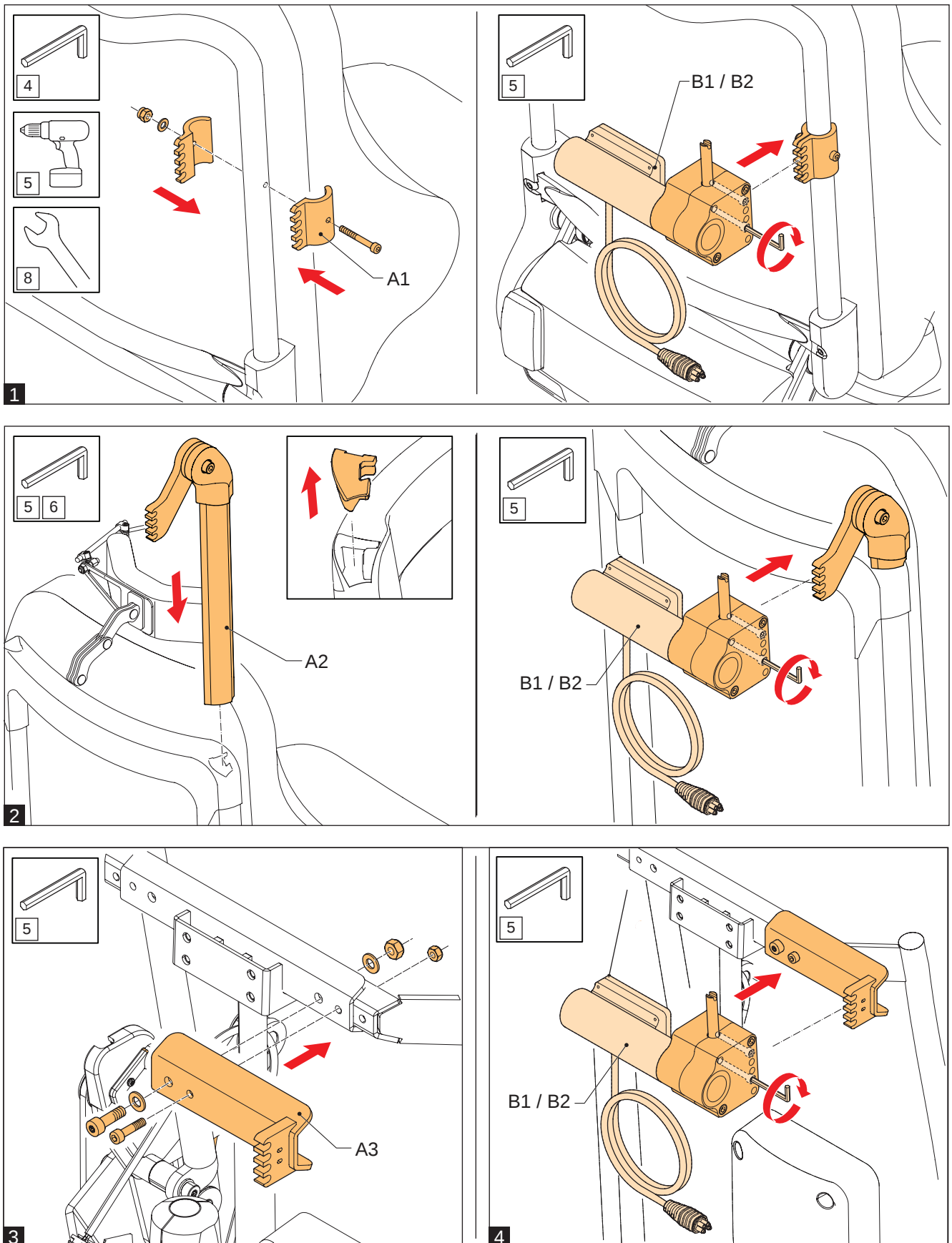


### 3.2.2 Chin Arm (assembly 2)

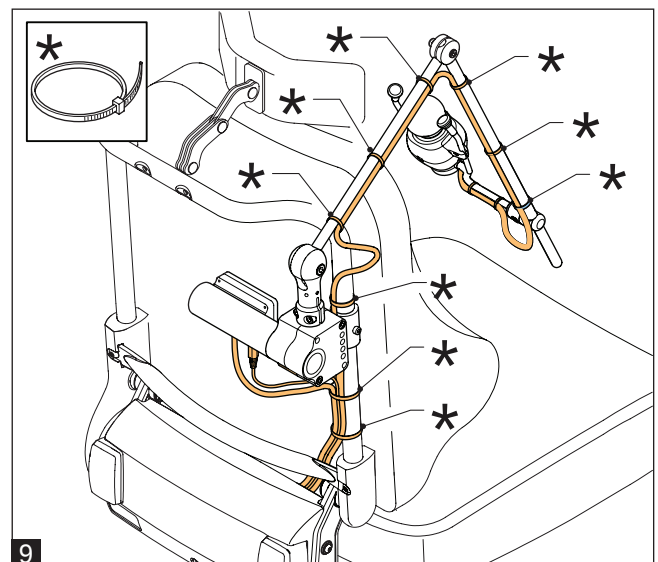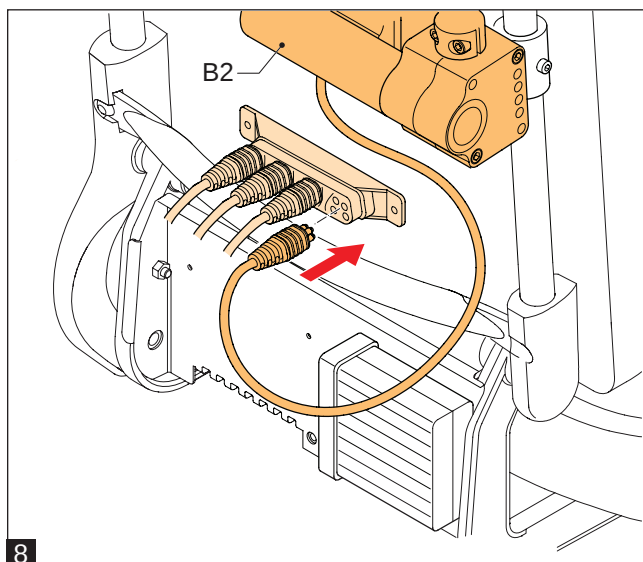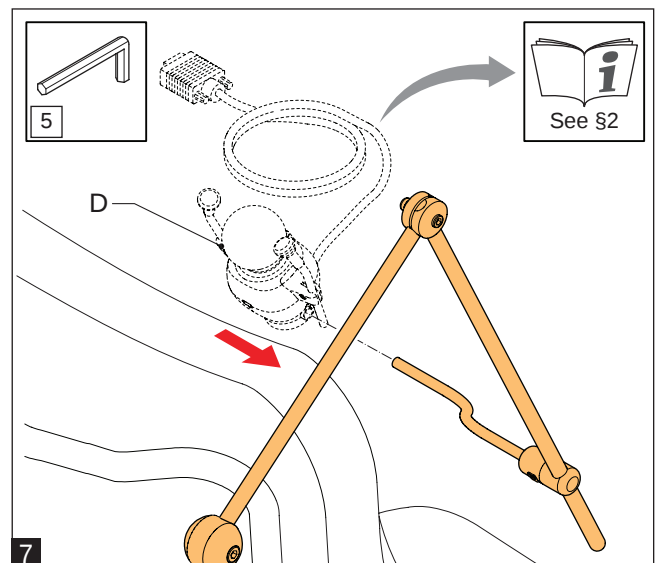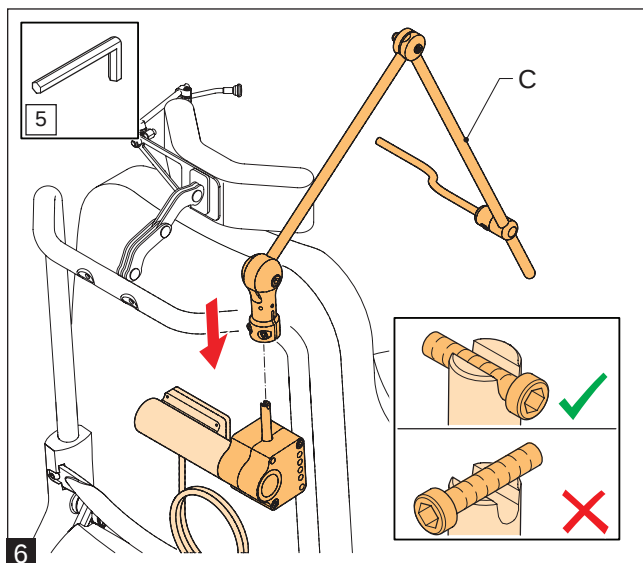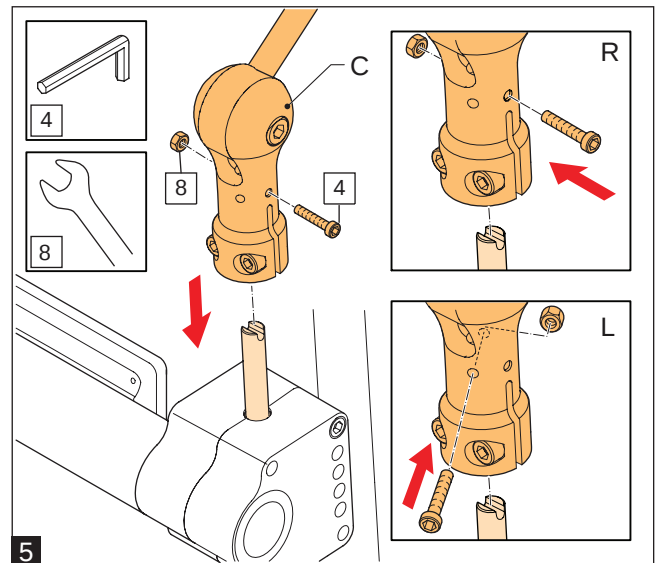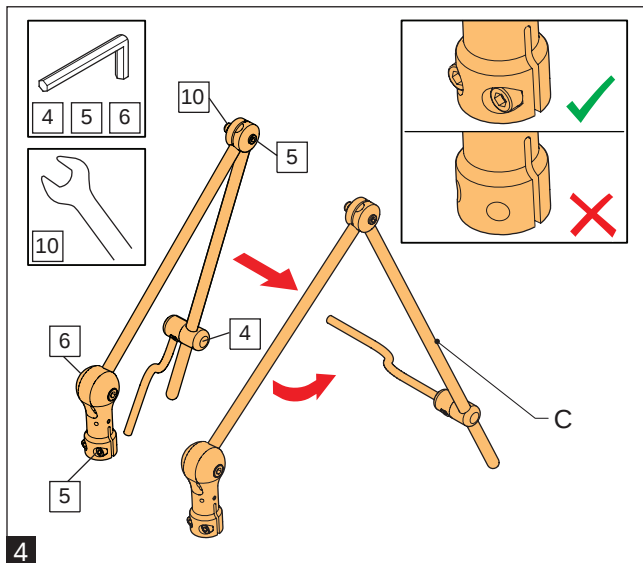


### 3.2.3 Chin Arm (assembly 2)

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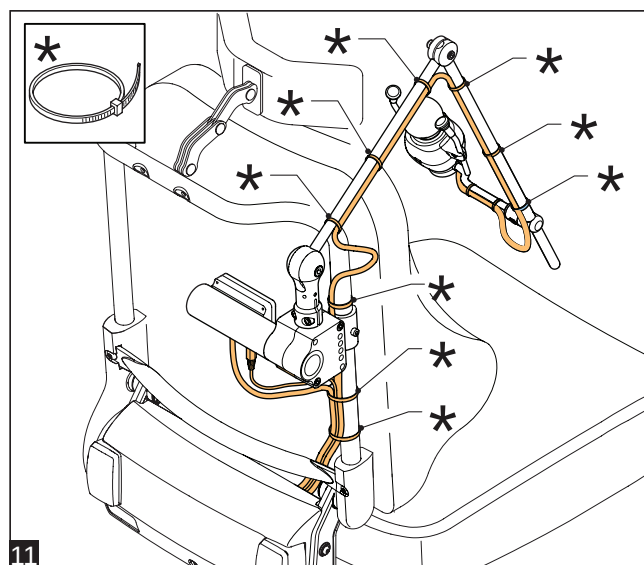
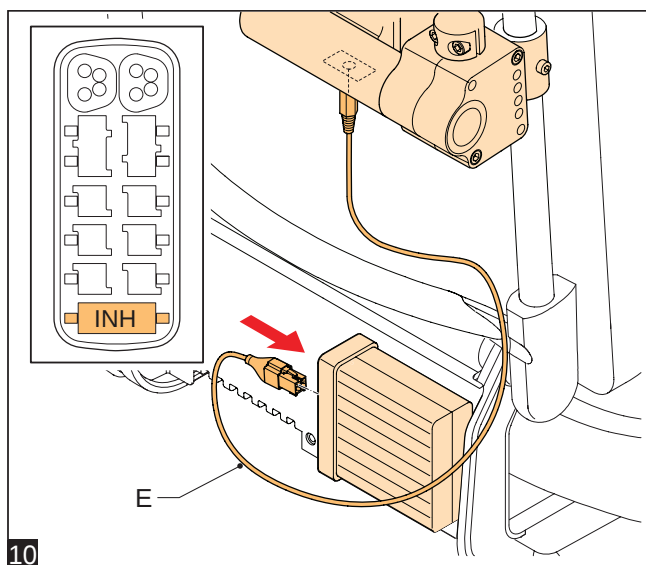


### 3.3 Chin Arm (adjustments)

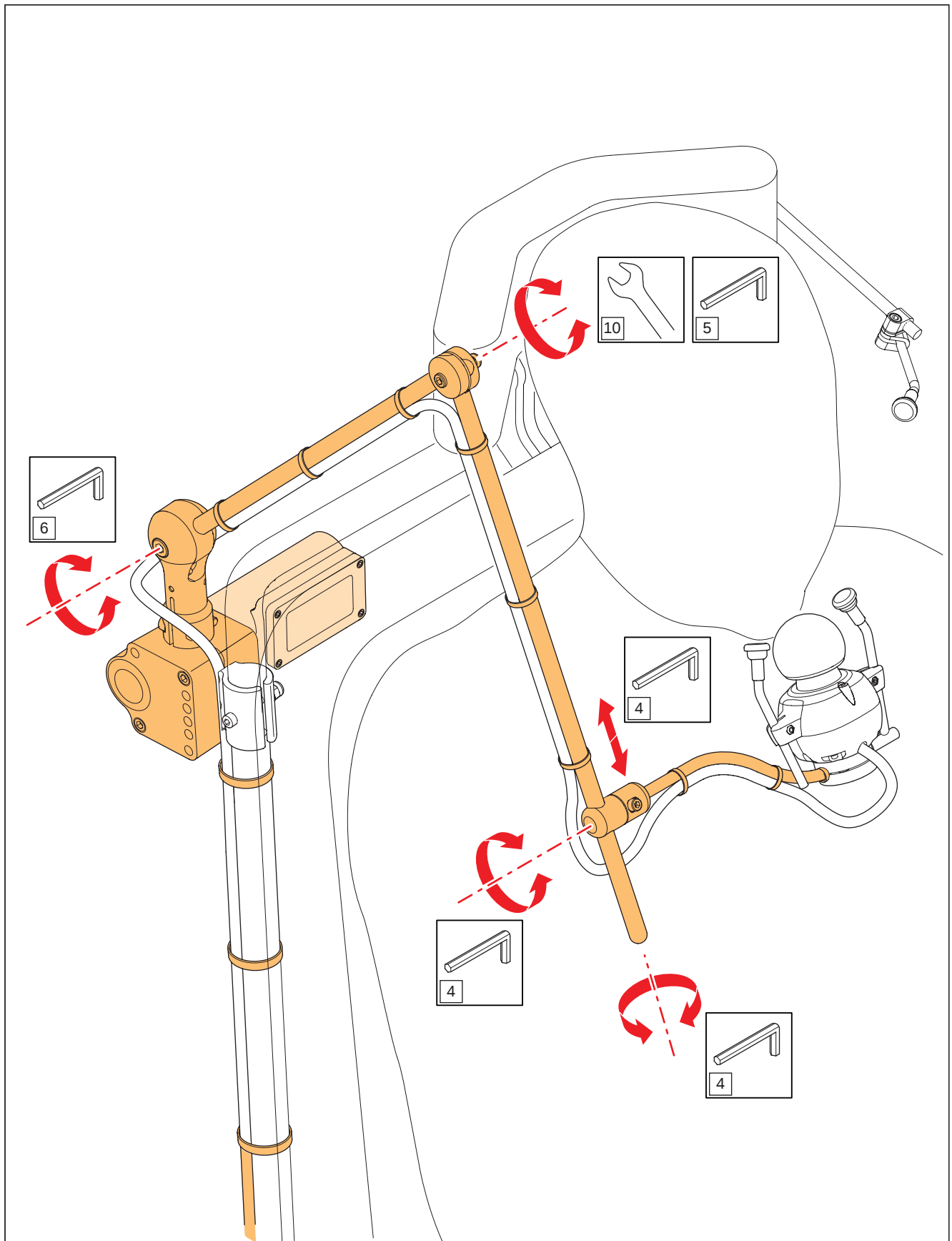


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







### 3.4 Headrest Switch

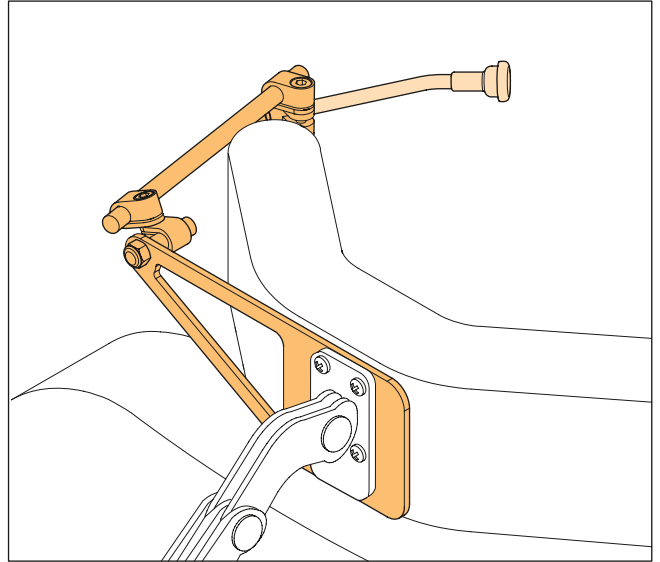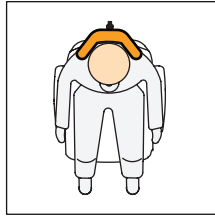


### 3.4.1 Headrest Switch (parts)

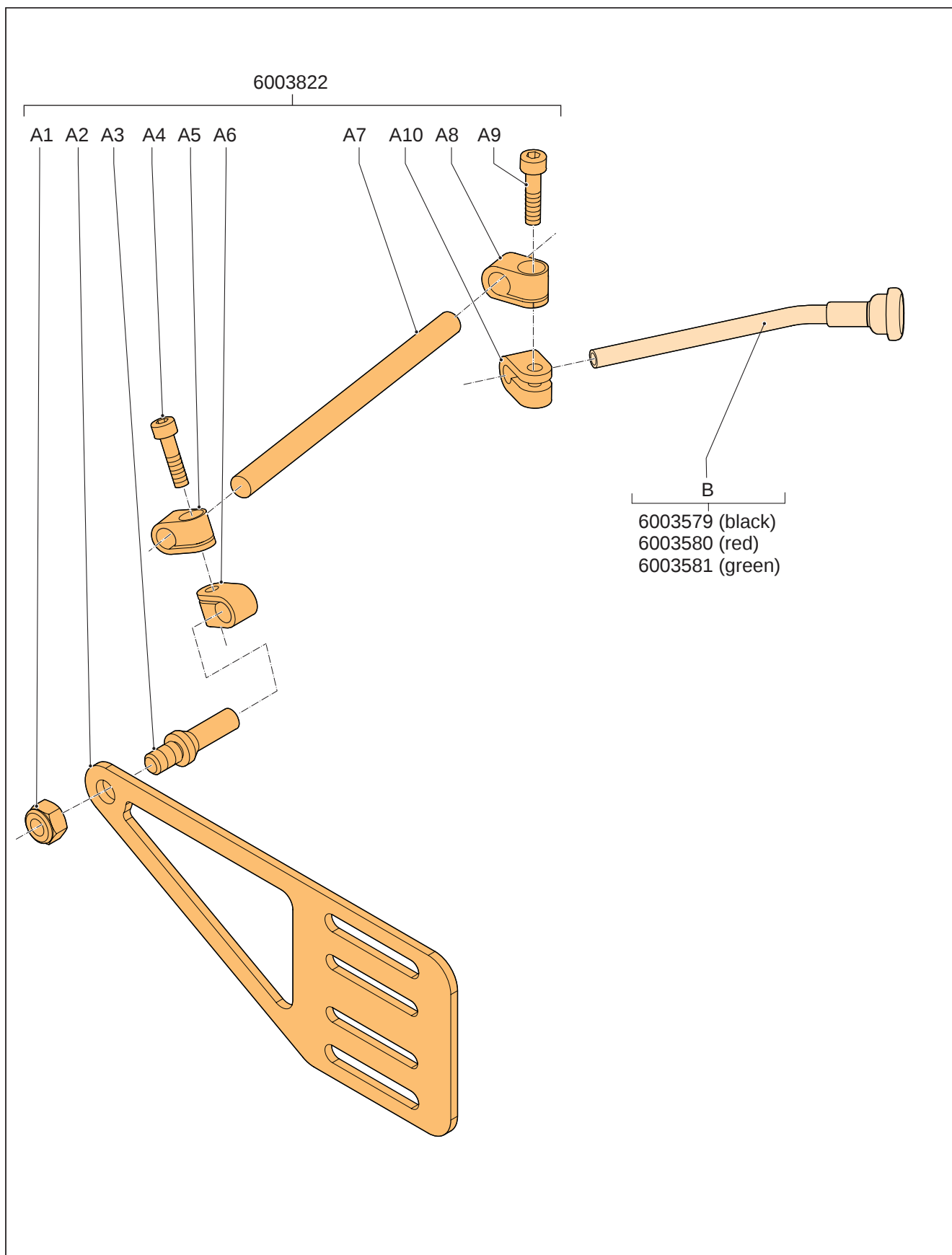


### 3.4.2 Headrest Switch (assembly)

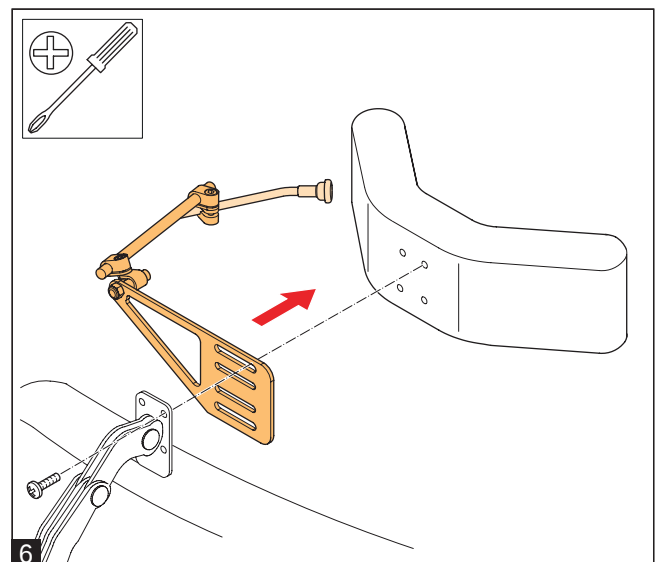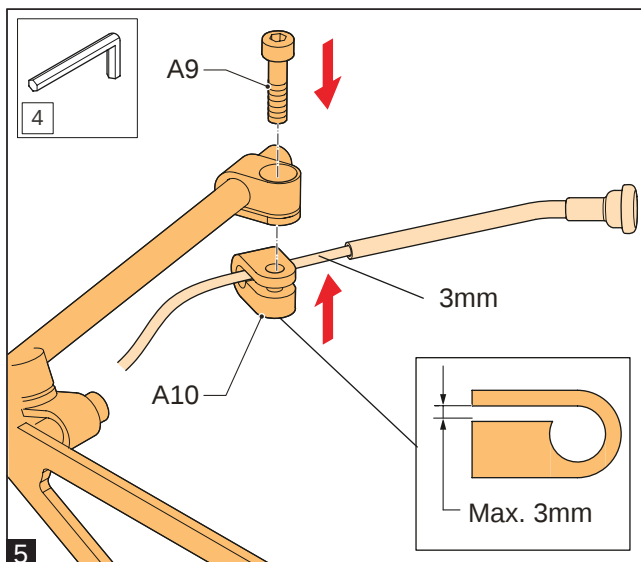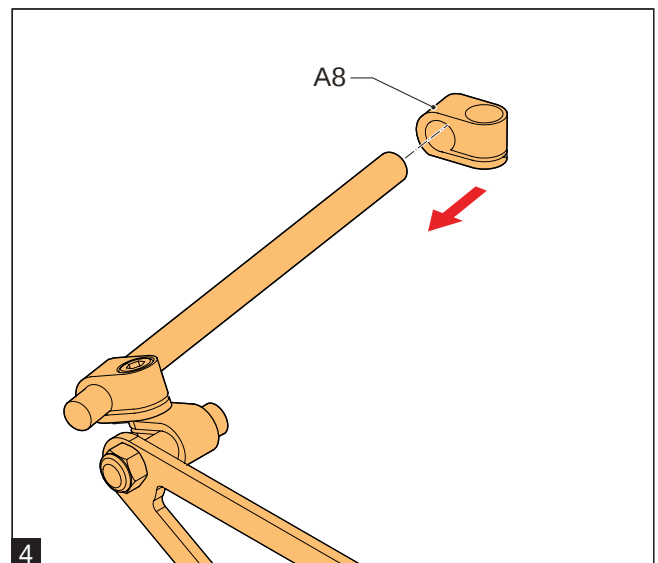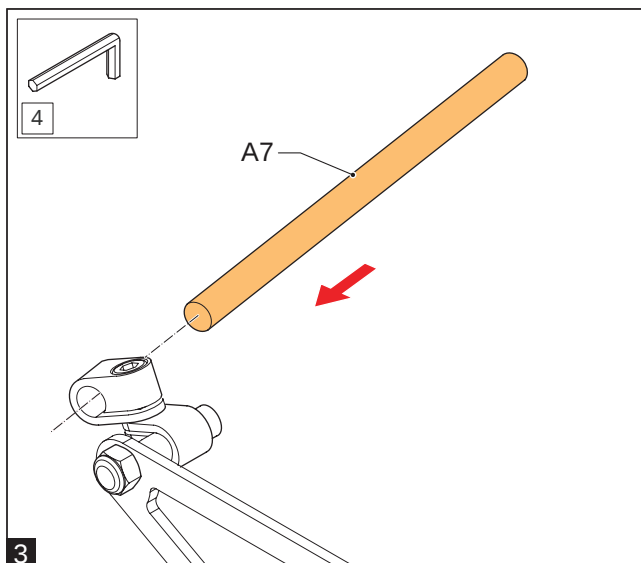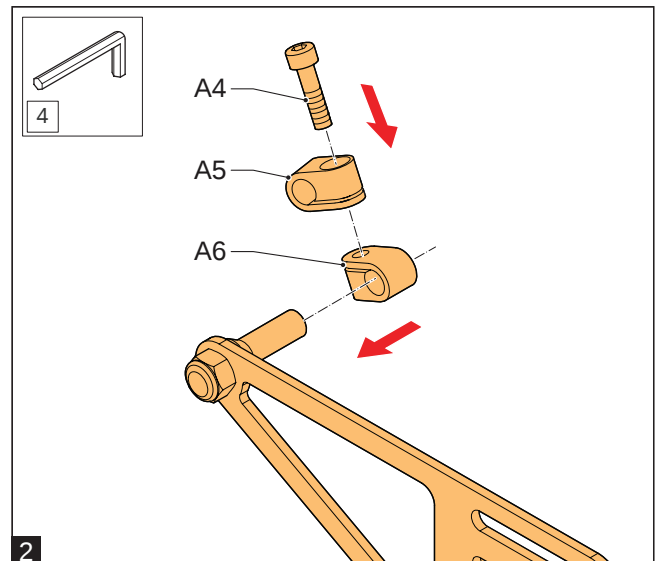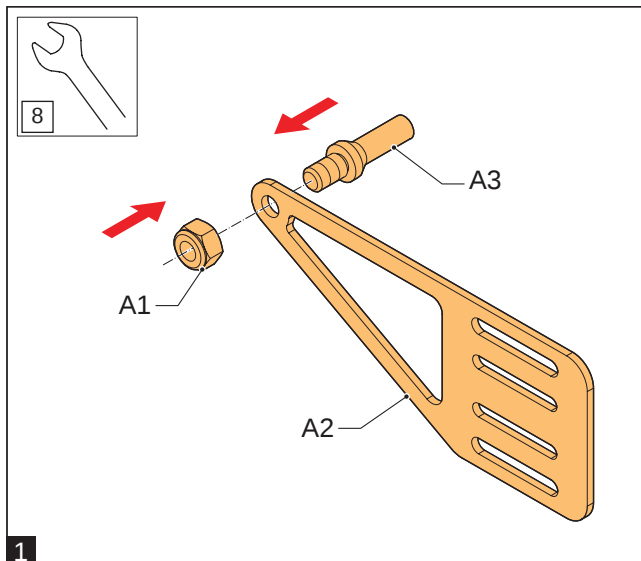

### 3.4.3 Headrest Switch (ombouw)

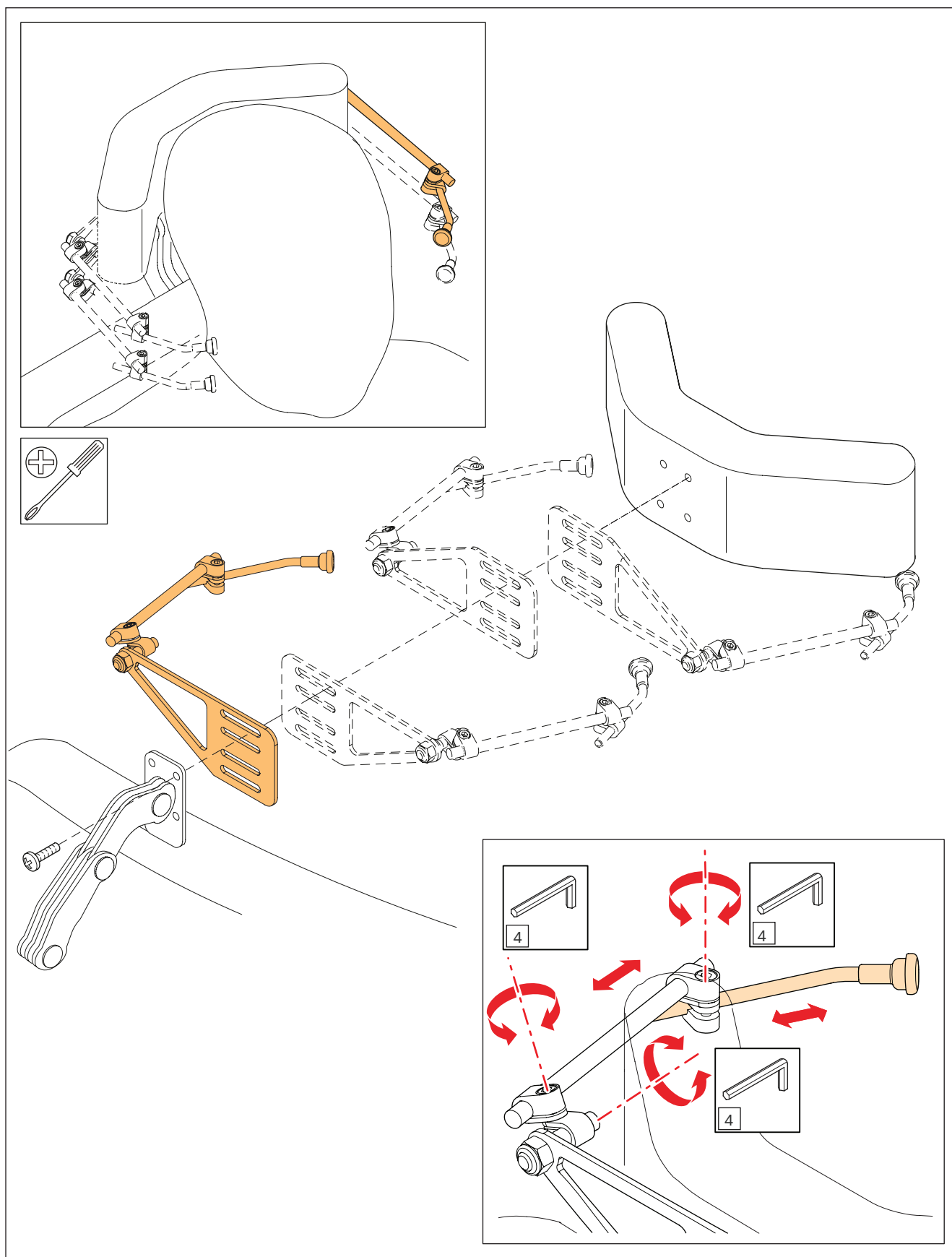

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



## MER ENN BARE PRODUKTER!

Våre lokale salgsrepresentanter hjelper deg gjerne med å vurdere alternative løsninger og tilpasse disse.

### SUNRISE MEDICAL KONTAKTINFORMASJON



#### Østfold, Buskerud

Stine Bukholm Tel. 93470066  
stine.bukholm@sunrisemedical.no

Tom Arild Johnsen Tel. 97628818  
tom.johnsen@sunrisemedical.no

#### Oslo, Akershus

Paul Ullerlien Tel. 90958355  
paul.ullerlien@sunrisemedical.no

Lars Marthinsen Tel. 99104450  
lars.marthinsen@sunrisemedical.no

Per Rune Molteberg Tel. 99095204  
per.molteberg@sunrisemedical.no

#### Hedmark, Oppland

Arnfinn Hellerud Tel. 91889260  
arnfinn.hellerud@sunrisemedical.no

Geir Birger Ekerhaugen Tel. 90136817  
geir.ekerhaugen@sunrisemedical.no

#### Vestfold, Telemark

Thor Arne Tvetene Tel. 90918570  
thor-arne.tvetene@sunrisemedical.no

#### Aust-Agder, Vest-Agder

Karl Rasmussen Tel. 97578003  
karl.rasmussen@sunrisemedical.no

#### Rogaland

Frode Lundal Tel. 91775110  
frode.lundal@sunrisemedical.no

#### Hordaland

Frode Gudvangen Tel. 48209891  
frode.gudvangen@sunrisemedical.no

#### Sogn og Fjordane

Jostein Slettehaug Tel. 99229766  
jostein.slettehaug@sunrisemedical.no

Frode Gudvangen Tel. 48209891  
frode.gudvangen@sunrisemedical.no

#### Møre og Romsdal

Jostein Slettehaug Tel. 99229766  
jostein.slettehaug@sunrisemedical.no

#### Sør-Trøndelag

Bjørn Tore Grande Tel. 91707330  
bjorn.grande@sunrisemedical.no

#### Nord-Trøndelag

Dag Håkon Edvardsen Tel. 48217313  
dag.edvardsen@sunrisemedical.no

#### Nordland

Uno Gjerstad Tel. 91880332  
uno.gjerstad@sunrisemedical.no

Dag Håkon Edvardsen Tel. 48217313  
dag.edvardsen@sunrisemedical.no

#### Troms, Finnmark

Uno Gjerstad Tel. 91880332  
uno.gjerstad@sunrisemedical.no

All informasjon kan endres uten varsel.  
Ta kontakt med Sunrise Medical om du har spørsmål. Dersom du er svaksynt, kan denne brosjyren ses i PDF-format på [www.SunriseMedical.no](http://www.SunriseMedical.no).



Sunrise Medical AS  
Delitoppen 3, 1540 Vestby

T: +47 66 96 38 00  
F: +47 66 96 38 80

E: [post@sunrisemedical.no](mailto:post@sunrisemedical.no)  
[www.SunriseMedical.no](http://www.SunriseMedical.no)