



Technical / News Bulletin

UK





BULLETIN NO.:
TB UK123
DATE ISSUED
29/07/2019

BULLETIN DETAILS

Product Group:	Seating		
Models Affected:	J3 Back Support Range		
Effective Date:	15 th July 2019	Serial No. Effective From:	N/A
Bulletin Subject:	Updates to J3 Back Supports		

Subject of Bulletin

As part of our continuing commitment to design improvement, we are introducing new innovations to our J3 backrest range to make it even more versatile. These include paediatric options, an expanded range of sizes, an extra deep contour option and full integration with Whitmyer head supports.

Feature	Benefit
<p>Extra Deep Contour options for Posterior</p> <ul style="list-style-type: none"> • Deep Contour (PDC) • Mid Deep Contour (MDC) backs   <p>Deep Contour depth = 150 mm</p>   <p>NEW: Extra Deep Contour = 180</p>	<p>Improved fit <i>Better fit for clients with deeper trunk profiles due to soft tissue or other factors influencing body shape</i></p> <p>Greater support <i>Meets the needs of clients who require more aggressive support than the standard 150mm deep contour depth.</i></p> <p>Two height options <i>Available on the popular Mid Thoracic Tall (38 cm) and Upper Thoracic Medium (46 cm) and heights</i></p> 

Expanded height options on J3 Mid-Deep Contour backs



Fits a wider range of clients

- 10 new back heights are now available for the most popular J3 MDC contour depth
- Heights now range from 17 to 61 cm

**J3, now for kids too!
New 31 and 36 cm Mid Deep Contour width options to meet paediatric needs**



Paediatric and small adult J3 Mid-Deep Contour Backs Widths

- NEW 31cm and 36 cm width options
- 31 cm size fits children up to 18cm chest width
- 36 cm fits children and small adults up to 28 cm chest width

Available bundled on the Salsa M2 Mini
New smaller J3 back options are now included in the Salsa M2 and M2 Mini order form for easy ordering.

Full integration of Whitmyer headrests as standard options



Whitmyer Headrest pads now standard on J3 Backs:

- Same Whitmyer mounting hardware, already fully compatible with Whitmyer pads
- Head support stays consistent, even when clients need to change the backrest
- Integrated into the J3 order form for ease of prescribing

Old style J3 headrest pads, being replaced:



Compact HD hardware conversion kit for UT (Upper Thoracic) J3 back heights



Heavy Duty (HD) Hardware

Compact Hardware



Low profile mounting for J3 UT height backs

- Current standard HD hardware provides a durable, adjustable option, suitable to most clients
- Compact hardware option allows fitting of J3 UT height backs on back posts with very limited space
- Integrated into J3 order form, for efficient ordering of all needed parts for the hardware conversion
- Suitable for clients weighing up to 136 kg, who do not have strong extensor tone or other patterns of heavy backrest use

J3 PDC (Posterior Deep Contour) backrest discontinuation



Increased ease of ordering

- Reduces complexity when ordering J3 backs
- Old PDC Lower Thoracic height is fully equivalent to the new MDC Lower Thoracic option
- Posterior Deep option is still available on JAY Easy backs
- Item numbers will be discontinued by 16th September
- Spares (foam, shell, cover) will continue to be available for 5 years

Literature Affected:

Website: Available on www.sunrisemedical.co.uk

Owners Manual: Available on www.sunrisemedical.co.uk

If you have any questions regarding this bulletin, please do not hesitate to contact:

Customer Services	Customer Support	Technical Service Centre	E-mail
Telephone:	+44 (0)845 605 66 88 Option 2	+44 (0)845 605 66 88 Option 3	enquiries@sunmed.co.uk
Fax:	+44 (0)845 605 66 89	+44 (0)845 605 66 89	help.technical@sunmed.co.uk

Please refer to www.sunrisemedical.co.uk, or contact your area sales manager for more information