

Omni 2 – Heavy Duty Seat

Retro Fitting Help Guide





RETRO FITTING HELP GUIDE

Parts Required:

Electronics

Omni 2 Kit – 20011242 (includes Input Section, Display Screen, 250cm connecting cable)

Backpack, Mounting kit and R-Net bus cable

• 20009167

Omni 2 Flex Mount – Inc. adapter plate

• 20009173

Channel Nuts and Bolts

- 014925 x 2 M5x12 Cap Head
- 000128100-ET M5 T-Nut



RETRO FITTING HELP GUIDE

Parts Required:











RETRO FITTING HELP GUIDE

Step 1

 Remove Omni 1 or retro fit the Omni flex mount to the armrest. All parts included in the kit.





RETRO FITTING HELP GUIDE

Step 2

Remove any cable tidy and disconnect R-Net Bus. Cable from the Omni 1 if removing.





RETRO FITTING HELP GUIDE

Step 3

Unscrew Omni 1 from the mounting bracket to remove.





RETRO FITTING HELP GUIDE

Step 4

Fix the Omni 2 adapter plate to the Omni 2 Display Module with 2 small bolts.








RETRO FITTING HELP GUIDE

Step 5

Fix the Omni 2 with adapter plate to the Omni mounting bracket with 2 bolts.

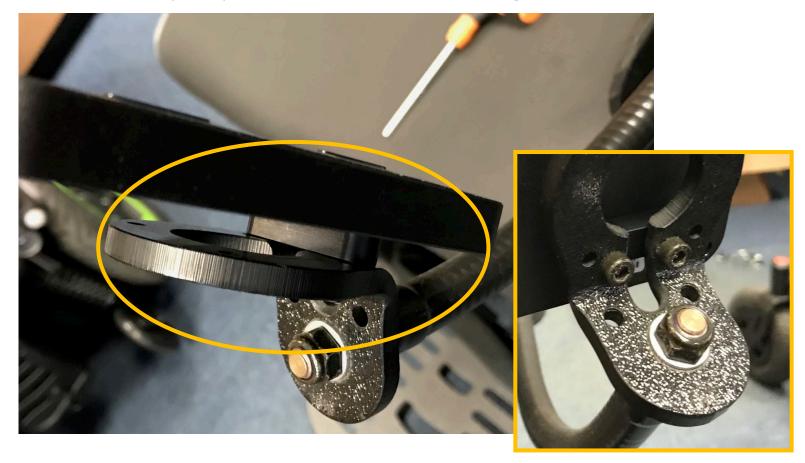




RETRO FITTING HELP GUIDE

Step 6

Take the 2 Omni 2 backpack mounting plates that fit together in this orientation.

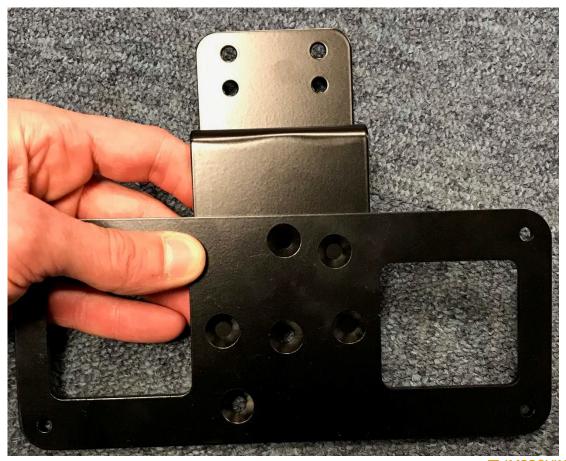



RETRO FITTING HELP GUIDE

Step 7

Fix the two brackets together using 2 counter sunk bolts, nuts and washers. You will need a 5mm Allen key and 10mm spanner.








RETRO FITTING HELP GUIDE

Step 8

The two brackets should look like this once fitted together correctly.

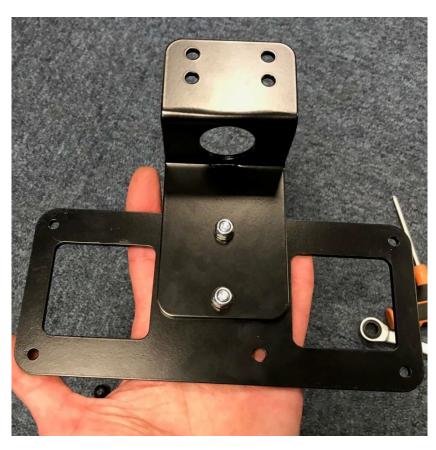





RETRO FITTING HELP GUIDE

Step 9

Open the Omni 2 backpack and drill holes in the orientation shown below.





RETRO FITTING HELP GUIDE

Step 10

Turn the Omni 2 backpack over and line up the mounting plate so that the holes

line up.






RETRO FITTING HELP GUIDE

Step 11

 Use a 8mm spanner and 5mm Allen key to fix the mounting plate to the backpack.



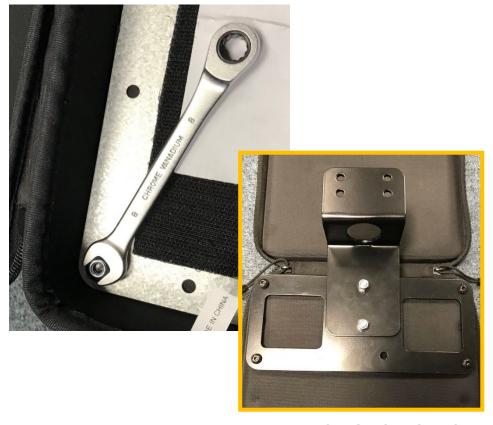




RETRO FITTING HELP GUIDE

Step 12

Remove the Jay Comfort backrest.

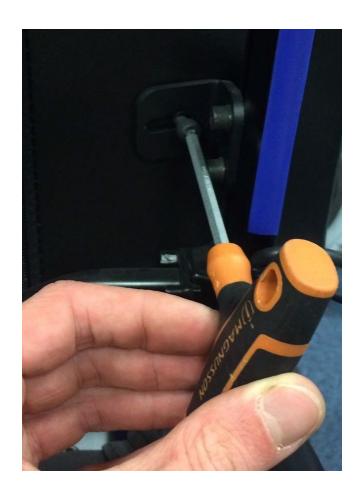





RETRO FITTING HELP GUIDE

Step 13

Backrest removed.






RETRO FITTING HELP GUIDE

Step 14

 Use a 5mm Allen key to remove the backrest tension bar. Remove both bolts from both sides but retain the bolt receiver blocks.



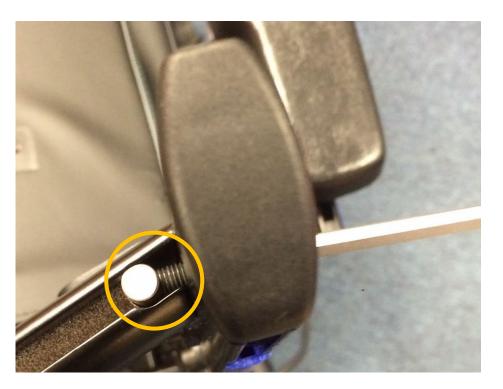



RETRO FITTING HELP GUIDE

Step 15

Retain these parts for when re-fitting.





RETRO FITTING HELP GUIDE

Step 16

 Remove the backrest tension bar and rotate it so that you can see the channel inside.





RETRO FITTING HELP GUIDE

Step 17

• Fix the M5 bolts into the channel nuts and slide into the channel in the backrest

tensioner.







RETRO FITTING HELP GUIDE

Step 18

Remove the M5 bolts from the channel nuts.





RETRO FITTING HELP GUIDE

Step 19

• Line up the Omni 2 backpack mounting bracket with the channel nuts and use the M5 bolts to secure centrally on the backrest tensioner bar.










RETRO FITTING HELP GUIDE

Step 20

• Tighten the bolts to secure to the backrest tensioner bar.

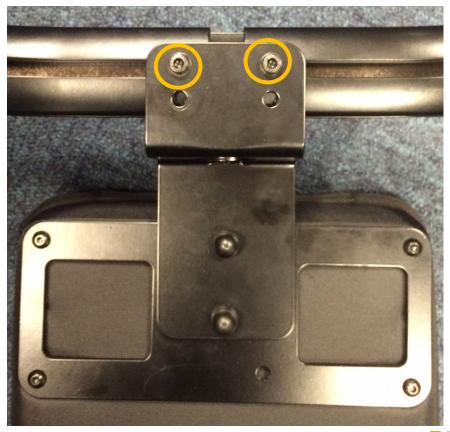



RETRO FITTING HELP GUIDE

Step 21

• Line up the backrest tensioner bar with the backposts.





RETRO FITTING HELP GUIDE

Step 22

• Secure the backrest tensioner back in place.










RETRO FITTING HELP GUIDE

Step 23

Secure the Jay Comfort backrest to the mounting brackets.

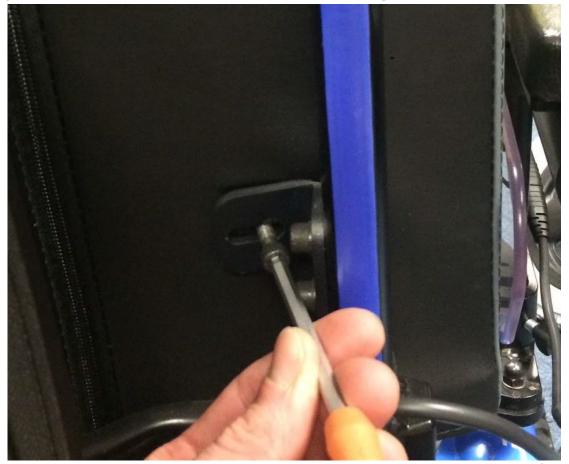



RETRO FITTING HELP GUIDE

Step 24

• Fit the Omni Flex mount bracket into the channels if required.






RETRO FITTING HELP GUIDE

Step 25

• Open the backpack and remove the Velcro pad.

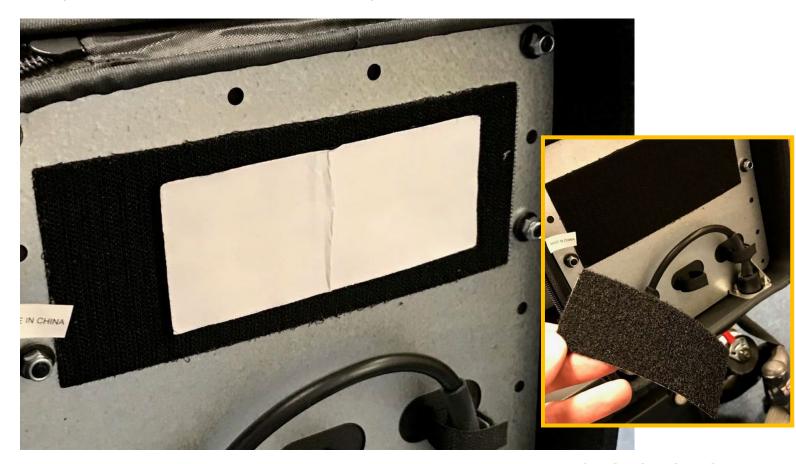



RETRO FITTING HELP GUIDE

Step 26

Take the Omni 2 Input Module and fix the Velcro pad to the back of the module.





RETRO FITTING HELP GUIDE

Step 27

• Fix the Input Module to the Velcro pad inside the backpack. Ensure you leave enough space for the R-Net bus cable.





RETRO FITTING HELP GUIDE

Step 28

 Plug in the charging loom into the port on the right hand side of the Input Module.







RETRO FITTING HELP GUIDE

Step 29

Plug in the charging loom into the port on the left hand side of the Input Module.





RETRO FITTING HELP GUIDE

Step 30

Connect the Omni 2 Input Module into the R-Net bus system.








RETRO FITTING HELP GUIDE

Step 30 cont.....

Connect the Omni 2 Input Module into the R-Net bus system.

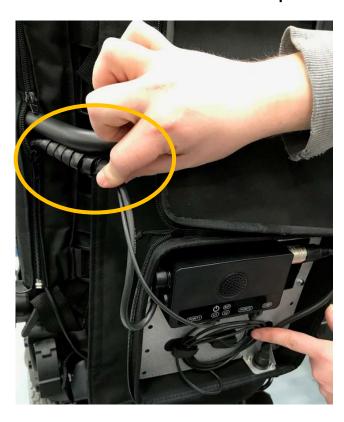





RETRO FITTING HELP GUIDE

Step 31

Ensure you use cable tidies to secure the Omni 2 connecting cable.

