WHEELCHAIR
ROLLSTUHL
FAUTEUIL ROULANT

Nitrum Nitrum Pro

LED LIGHT RETROFIT

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INSTALLATION DE LUMIÈRES
LED

The logo for Quickie, featuring a stylized swoosh above the word 'QUICKIE' in a bold, sans-serif font.

66009910

LED light retrofit-kit for Sopur/ Quickie Nitrum

The LED lights are mounted to the caster-links using an adhesive tape and an allen screw. The Calf-strap is designed as cable-guide and reinforced to bear the weight of the Controller Box.

The retrofit-kit for the LED-lights includes:

- Two self-tapping screws
- LED light incl. cable left hand side
- LED light incl. cable right hand side
- Controller.Box incl. Battery
- Reinforced Calf-strap
- 2 Velcro Zip-ties

Mounting Instructions:

1. Clean and degrease the bottom side of the caster-links on both sides of the chair
2. Pull the protective layer off the double adhesive tape on the bottom of the left-hand-side LED light
3. Insert the self-tapping screw into the bore on the LED-housing, and align the LED-housing on the caster-link (without already sticking the adhesive tape on the caster-link) so that the screw is aligned with the bore on the caster-link. The LEDs are facing towards the front of the wheelchair.
4. Now press the adhesive tape onto the caster-link to connect the LED-housing with the caster link. Press and hold for 20 seconds
5. Tighten the self-tapping screw with a max. torque of 2Nm (Fig. 1)
6. Repeat above for the right-hand-side LED light
7. Mount the calf-strap and adjust length as necessary
8. Route the cables of the LED lights (A) along the frame towards the calf-strap. Use Velcro-zip-ties (B) to tighten the cables to the frame underneath the calf-strap (Fig.2)
9. Route the cables through the Velcro-channel (C) on the backside of the calf-strap towards the middle of the calf strap (Fig.2)
10. Clip the Controller-Box on the calf-strap, so that it's located in the middle of the calf-strap. The USB connector is facing towards the front of the wheelchair
11. Insert the LED-cable pins into the connectors on each side of the Controller-Box
12. Cut the rear-cover of the calf-strap to length and Velcro it to the rear-side of the calf-strap so that it covers the cables and the Controller-Box

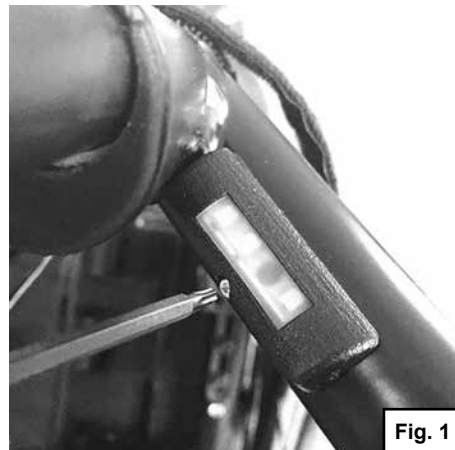
Fig. 1

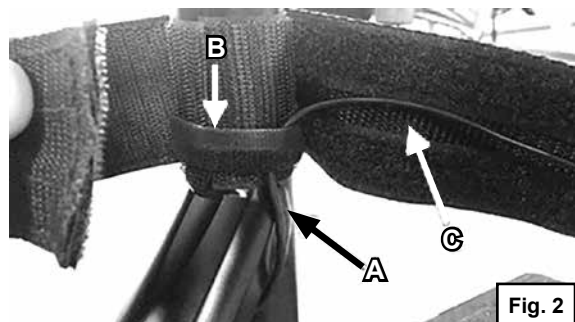
Fig. 2

Instructions for Operation

Switching the lights on: by pushing the button A on top of the control-Box (Fig.3), you can cycle through operating-modes of the LEDs:

1. Full power mode
2. Low-power mode
3. Flash-mode
4. Off

The charging-status of the battery is indicated by the backlit button A :

- red: charging-status <50%
- red (when battery is connected to the charger): battery is charging
- green (when battery is connected to the charger): battery fully charged

Remove the control-box from the calf-strap for charging: detach cables on the sides by gently pulling the pins out of the connectors (C). Now pull the control-box upwards to remove it from the calf-strap.

Charge the battery: connect the USB-cable to the connector (B) on the front-side of the control-Box. Use a standard USB-charger or your Computer's USB-port to charge the battery. Green light will indicate a fully charged battery.

Battery Specification:


Battery-type	Lithium-Ion
Nominal capacity	2600mAh
Nominal voltage	3.6V
Charging time	3 hours

WARNING

- Before you use the LED light for the first time, please make sure to fully charge the battery
- The Control-Box contains a Lithium-Ion battery, which is a hazardous product.
- Electrical and electronic equipment need to be disposed of separately to general household waste at specific state provided locations. The correct disposal and separate collection of used appliances serves to prevent potential damage to health and the environment. It is a requirement for the re-utilisation and recycling of used electrical and electronic equipment.
- Detailed information on the disposal of your used equipment can be obtained from your local authority, your waste disposal service, the specialist dealer from which you purchased the product, or your sales contact.

Fig. 3

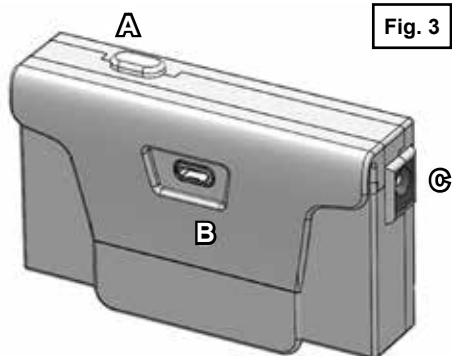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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S